




PRESS
X 3

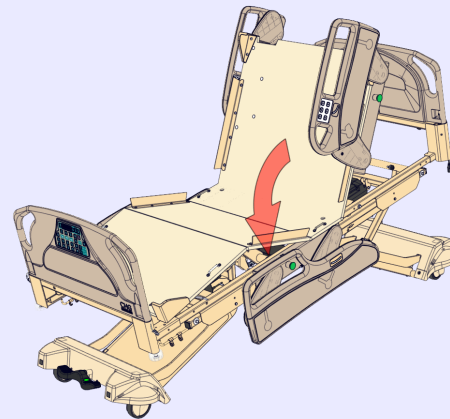

1. Electronic Foot Siderail Release

The Electronic Foot Siderail Release feature allows patients to have a greater level of mobility and independence. This feature allows the patient to unlock/unlatch the foot siderails while in the bed by pressing a button on the head siderail control. This will permit patient ingress to and egress from the bed without the aid from staff.

2. Enhanced Siderail Controls


The head siderails incorporate integrated controls which offer both staff and patient control of basic bed operations:

- Head UP/DOWN
- Bed UP/DOWN
- Knee UP/DOWN
- Nurse Call Feature
- Electronic Foot Siderail Release

IMPORTANT: Bed operation from the siderail control is disabled when the patient lock-outs are activated. However, the Nurse Call feature always remains active.

3. Electronic Foot Siderail Release Functionality

To avoid accidental release of the foot siderail, the release button must be pressed 3 times to activate the electronic release.

- Press the  "FOOT RAIL/PRESS 3X" button on the desired siderail 3 times within a period of 5 seconds. An audible chirp will sound after each button press as confirmation to the patient. An audible "click" should be heard when the foot siderail electronically unlocks/unlatches.

4. To Enable/Disable the Electronic Foot Siderail Release Feature

From the SERVICE Menu:

1. Press the UP/DOWN arrow buttons to scroll to the FOOT RAIL RELEASE screen
2. Press the ON/ENTER button to enter this menu option. The current selection will be shown
 - LEFT ENABLED (patient left foot siderail release enabled)
 - RIGHT ENABLED (patient right foot siderail release enabled)
 - BOTH ENABLED (both patient left and patient right foot siderail release enabled)
 - RAIL RELEASE DISABLED
3. Press the UP/DOWN arrow buttons to scroll between selection options
4. Press the ON/ENTER button to make desired selection

An audible chirp will sound to indicate that the Electronic Foot Siderail Release setting has been changed. Press the 0.0/ZERO button to "escape" to the home screen.



FOR REFERENCE ONLY, PLEASE REFER TO MANUAL FOR FUTURE INSTRUCTIONS