

stryker[®]
Medical

*2035/2036 **APEX**[™] Critical Care Bed*

Maintenance Manual

For Beds with serial numbers of 9806035296 and above.

For Parts or Technical Assistance
800-327-0770

INTRODUCTION AND SET-UP INFORMATION

Introduction 1-1

Specifications 1-1

Warning/Caution/Note Information 1-1

Warranty 1-2, 1-3

Safety Tips and Guidelines 1-4, 1-5

Set-Up Procedures 1-6

Bed Symbols 1-7 – 1-14

PREVENTIVE MAINTENANCE

Cleaning 2-2

Preventive Maintenance Checklist 2-3

General Information 2-4

TROUBLESHOOTING

Troubleshooting Guide 3-2

Optional Dynamic Mattress System Troubleshooting Guide 3-3

ELECTRICAL SYSTEM INFORMATION

CPU Board 4-2, 4-3

Power Supply 4-4

Optional DMS Control Board 4-5

Optional StryKair Power Supply 4-6

Optional Smart TV Circuit Board 4-7, 4-8

Optional Bed Communications Tester 4-9

Head Wall Output Configuration 4-10

QUICK REFERENCE REPLACEMENT PARTS LIST

Quick Reference Replacement Parts List 5-1

BASE MAINTENANCE

Static Discharge Precautions 6-2

Checking Hydraulic Fluid Level 6-3

BASE MAINTENANCE (CONTINUED)

Bleeding Air From The Hydraulic System 6-3

Jack Descent Rate Adjustment 6-4

Caster Assembly Replacement 6-4

Caster Cover Installation And Removal 6-5

Steer Caster Linkage Adjustment 6-6

Brake Adjustment 6-7

110V Power Cord Replacement 6-7

Lift Potentiometer Replacement 6-8

Lift Potentiometer "Burn-In" Procedure 6-9

Coil And Wire Replacement 6-10

Manual Release Valve Replacement 6-11

Solenoid Release Valve Replacement 6-11

Hydraulic Pump And Motor Replacement 6-12

Jack Replacement 6-13

LITTER MAINTENANCE

Scale System Diagnostics and Calibration 7-2 – 7-4

Load Cell Replacement 7-4

Head Motor Removal and Replacement 7-5

Knee Motor Removal and Replacement 7-6

Power Supply Removal and Replacement 7-7

CPU Board Removal and Replacement 7-7

Fowler and Lift Potentiometer "Burn-In" Procedure 7-8

SIDERAIL MAINTENANCE

Head and Foot End Siderail Cover Removal 8-2

Head and Foot Molded Siderail Replacement 8-3

Head End Siderail Cable Replacement 8-4, 8-5

FOOT BOARD MAINTENANCE

Foot Board Hinge Removal 9-2

Foot Board Module Replacement 9-3

Foot Board Interface Plug Replacement 9-4

DYNAMIC MATTRESS SYSTEM MAINTENANCE

Mattress Cover Replacement 10-2

Air Bladder Replacement 10-3

Foam Base Assembly Replacement 10-3

Control Unit Circuit Board Replacement 10-4

Control Unit Air Compressor Replacement 10-4, 10-5

Control Unit Replacement 10-5

ASSEMBLY DRAWINGS AND PARTS LISTS

Base Assembly 11-3 – 11-14

Brake Adjuster Assembly 11-15

Brake Cam Assembly 11-16

Caster Assembly 11-17

Steer Caster Assembly 11-18

Wheel Assembly 11-19

Fifth Wheel Arm Assembly 11-20

Jack/Motor Assembly 11-22, 11-23

Jack Assembly 11-24, 11-25

Jack Base Assembly 11-26

Jack Pump Piston Assembly 11-27

Rack and Pinion Assembly 11-28, 11-29

Litter Assembly 11-30 – 11-42

Head End Siderail Assembly 11-43– 11-57

Head End Siderail Timing Link Assembly 11-58, 11-59

Head End Siderail Inner Panel Assembly 11-60

Head End Siderail Outer Panel Assembly 11-61

ASSEMBLY DRAWINGS AND PARTS LISTS (CONTINUED)

Head End Siderail DMS Module Assembly	11-62
Head End Siderail Smart TV Module Assembly	11-63, 11-64
Head End Siderail TV Control Module Assembly	11-65, 11-66
Foot End Siderail Assembly	11-67 – 11-70
Foot End Siderail Timing Link Assembly	11-71, 11-72
Head Board Assembly	11-73
Foot Board Assembly	11-74 – 11-79
Foot Board Main Module Assembly	11-80
Foot Board CPR Drop/Cardiac Chair Module Assembly	11-81
Foot Board Bed Exit Module Assembly	11-82
Pendant Assembly	11-83
Removable I.V. Pole Assembly	11-84
2-Stage I.V. Mounting Assembly, Foot End	11-85
2-Stage I.V. Mounting Assembly, Head End	11-86
2-Stage I.V. Mounting Assembly, Dual Head End	11-87
2-Stage I.V. Pole Assembly	11-88
I.V. Pole Latch Assembly	11-89
Fowler X-Ray Cassette Holder Assembly	11-90
Siderail Transducer Mount Assembly	11-91
I.V. Pole Transducer Mount Assembly	11-92
Upright Oxygen Bottle Holder Assembly	11-93
Defibrillator Tray Assembly	11-94
Optional Bed Extender Pad	11-95

INTRODUCTION

This manual is designed to assist you with the operation of the Model 2035 Stryker **APEX** Critical Care Bed. Read it thoroughly before using the equipment.

SPECIFICATIONS

Maximum Weight Capacity	500 pounds or 227 kilograms
Weigh System Capacity (optional equipment)	patients weighing up to 500 pounds or patients weighing up to 227 kilograms
Weigh System Accuracy (optional equipment)	± 1% of total patient weight
Overall Bed Length/Width	L-91" /W-42.5" or L-231 cm /W-108 cm
Minimum/Maximum Bed Height	20 1/2" to 31" (22 1/2" to 35" enhanced fluoro option) or 52 cm to 79 cm (57 cm to 89 cm enhanced fluoro option)
Knee Gatch Angle	0° to 35°
Back Angle	0° to 90°
Trendelenburg/Reverse Trendelenburg	-10° to +10° (standard height) -12° to +12° (enhanced fluoro option)
Electrical Requirements	115 VAC, 60 Hz, 7.0 Amps 115 VAC, 60 HZ, 12.0 Amps (with optional 115V outlet)
Noise Level	> 65 Decibels

Stryker reserves the right to change specifications without notice.

WARNING / CAUTION / NOTE DEFINITION

The words WARNING, CAUTION and NOTE carry special meanings and should be carefully reviewed.

WARNING

The personal safety of the patient or user may be involved. Disregarding this information could result in injury to the patient or user.

CAUTION

These instructions point out special procedures or precautions that must be followed to avoid damaging the equipment.

NOTE

This provides special information to make important instructions clearer.

WARNING

Always apply the caster brakes when a patient is getting on or off the bed. Push on the bed to ensure the brakes are securely locked. Always engage the brakes unless the bed is being moved. Injury could result if the bed moves while a patient is getting on or off the bed.

Warranty

Limited Warranty:

Stryker Medical Division, a division of Stryker Corporation, warrants to the original purchaser that its products should be free from defects in material and workmanship for a period of one (1) year after date of delivery. Stryker's obligation under this warranty is expressly limited to supplying replacement parts and labor for, or replacing, at its option, any product which is, in the sole discretion of Stryker, found to be defective. Stryker warrants to the original purchaser that the frame and welds on its beds will be free from structural defects for as long as the original purchaser owns the bed. If requested by Stryker, products or parts for which a warranty claim is made shall be returned prepaid to Stryker's factory. Any improper use or any alteration or repair by others in such manner as in Stryker's judgement affects the product materially and adversely shall void this warranty. No employee or representative of Stryker is authorized to change this warranty in any way.

This statement constitutes Stryker's entire warranty with respect to the aforesaid equipment. STRYKER MAKES NO OTHER WARRANTY OR REPRESENTATION, EITHER EXPRESSED OR IMPLIED, EXCEPT AS SET FORTH HEREIN. THERE IS NO WARRANTY OF MERCHANTABILITY AND THERE ARE NO WARRANTIES OF FITNESS FOR ANY PARTICULAR PURPOSE. IN NO EVENT SHALL STRYKER BE LIABLE HEREUNDER FOR INCIDENTAL OR CONSEQUENTIAL DAMAGES ARISING FROM OR IN ANY MANNER RELATED TO SALES OR USE OF ANY SUCH EQUIPMENT.

To Obtain Parts and Service:

Stryker products are supported by a nationwide network of dedicated Stryker Field Service Representatives. These representatives are factory trained, available locally, and carry a substantial spare parts inventory to minimize repair time. Simply call your local representative, or call Stryker Customer Service at (800) 327-0770.

Supplemental Warranty Coverage:

Stryker has developed a comprehensive program of extended warranty options designed to keep your equipment operating at peak performance at the same time it eliminates unexpected costs. We recommend that these programs be activated *before* the expiration of the new product warranty to eliminate the potential of additional equipment upgrade charges. Stryker offers the following Supplemental Warranties:

Extended (Parts and Labor)

- All replacement parts (excluding mattresses and consumable items)
- Labor and travel for *all* scheduled and unscheduled calls
- Annual Preventive Maintenance Inspections and repairs
- JCAHO paperwork for preventive maintenance
- Priority Emergency Service

Standard (Labor Only):

- Labor and travel for *all* scheduled and unscheduled calls
- Annual Preventive Maintenance Inspections and repairs
- JCAHO paperwork for preventive maintenance
- Priority Emergency Service

Basic (Parts Only):

- All replacement parts (excluding mattresses and consumable items)
- Priority Emergency Service

Please call your local representative, or call (800) 327-0770 for further information

Warranty

Return Authorization:

Merchandise cannot be returned without approval from the Stryker Customer Service Department. An authorization number will be provided which must be printed on the returned merchandise. Stryker reserves the right to charge shipping and restocking fees on returned items.

SPECIAL, MODIFIED, OR DISCONTINUED ITEMS NOT SUBJECT TO RETURN.

Damaged Merchandise:

ICC Regulations require that claims for damaged merchandise must be made with the carrier within fifteen (15) days of receipt of merchandise. **DO NOT ACCEPT DAMAGED SHIPMENTS UNLESS SUCH DAMAGE IS NOTED ON THE DELIVERY RECEIPT AT THE TIME OF RECEIPT.** Upon prompt notification, Stryker will file a freight claim with the appropriate carrier for damages incurred. Claim will be limited in amount to the actual replacement cost. In the event that this information is not received by Stryker within the fifteen (15) day period following the delivery of the merchandise, or the damage was not noted on the delivery receipt at the time of receipt, the customer will be responsible for payment of the original invoice in full.

Claims for any short shipment must be made within thirty (30) days of invoice.

International Warranty Clause:

This warranty reflects U.S. domestic policy. Warranty outside the U.S. may vary by country. Please contact your local Stryker Medical representative for additional information.

Safety Tips and Guidelines

Before operating the 2035, it is important to read and understand all information in this manual. Carefully read and strictly follow the safety guidelines listed on this page.

It is important that all users have been trained and educated on the inherent hazards associated with the usage of electric beds.



WARNING

- The 2035 is not intended for use with patients less than two years of age.
- Powered bed mechanisms can cause serious injury. Operate bed only when all persons are clear of the mechanisms.
- To help reduce the number and severity of falls by patients, always leave the bed in the lowest position when the patient is unattended.
- Leave the siderails fully up and locked when the patient is unattended. When raising the siderails, listen for the "click" that indicates the siderail has locked in the up position. Pull firmly on the siderail to ensure it is locked into position.
Siderails are not intended to be a patient restraint device. It is the responsibility of the attending medical personnel to determine the degree of restraint necessary to ensure a patient will remain safely in bed.
- Always keep the caster brakes applied when a patient is on the bed (except during transport). Serious injury could result if the bed moves while a patient is getting in or out of bed. After the brake pedal is applied, push on the bed to ensure the brakes are locked. When moving the bed, toggle the steer pedal to put the bed in the steer mode. This locks the swivel motion of the right foot end caster and makes the bed easier to move.
- When large spills occur in the area of the circuit boards, 110 volt cables and motors, immediately unplug the bed power cord from the wall socket. Remove the patient from the bed and clean up the fluid. Have maintenance completely check the bed. Fluids can affect the operational capabilities of any electrical product. **DO NOT** put the bed back into service until it is completely dry and has been thoroughly tested for safe operation.
- Do not steam clean or hose off the bed. Do not immerse any part of the bed. The internal electric parts may be damaged by exposure to water. Hand wash all surfaces of the bed with warm water and mild detergent. Dry thoroughly. Quaternary Germicidal Disinfectants, used as directed, and/or Chlorine Bleach products, typically 5.25% Sodium Hypochlorite in dilutions ranging between 1 part bleach to 100 parts water, and 2 parts bleach to 100 parts water are **not** considered mild detergents. **THESE PRODUCTS ARE CORROSIVE IN NATURE AND MAY CAUSE DAMAGE TO YOUR BED IF USED IMPROPERLY.** If these types of products are used to clean Stryker patient care equipment, measures must be taken to insure the beds are wiped with clean water and thoroughly dried following cleaning. Failure to properly rinse and dry the beds will leave a corrosive residue on the surface of the bed, possibly causing premature corrosion of critical components. Failure to follow the above directions when using these types of cleaners may void this product's warranty.
Clean Velcro **AFTER EACH USE**. Saturate Velcro with disinfectant and allow disinfectant to evaporate. (Appropriate disinfectant for nylon Velcro should be determined by the hospital.)
- Preventative maintenance should be performed at a minimum of biannually to ensure all bed features are functioning properly. Close attention should be given to safety features including, but not limited to: safety side latching mechanisms, frayed electrical cords and components, all electrical controls return to off or neutral position when released, caster braking systems, no controls or cabling entangled in bed mechanisms, leakage current 100 MA maximum, scale and bed exit systems calibrated properly.
- Always unplug bed during service or cleaning. When working under the bed with the bed in the high position, always place blocks under the litter frame and set the brakes to prevent injury in case the Bed Down switch is accidentally pressed.
- Explosion Hazard – do not use bed in the presence of flammable anesthetics.

Safety Tips and Guidelines



WARNING

Potential pinch points

Set-Up Procedures

It is important that the 2035 is working properly before it is put into service. The following list will help ensure that each part of the bed is tested.

- Plug the bed into a properly grounded, hospital grade wall receptacle and ensure the "Power" LED light at the foot end of the bed comes on.

 **WARNING**

The 2035 is equipped with a hospital grade plug for protection against shock hazard. It must be plugged directly into a properly grounded three-prong receptacle. Grounding reliability can be achieved only when a hospital grade receptacle is used.

-
- Plug the optional interface cable into the 37 pin connector under the litter frame at the head end of the bed, and into the "Patient Station", "Head Wall", "Docking Station", or equivalent (whichever applies). Test the interface cable to verify it is functioning properly.

 **WARNING**

Use only a Stryker supplied interface cable. Use of any other cable may cause the bed to function improperly which may result in patient or user injury.

-
- Ensure the siderails raise, lower and store smoothly and lock in the up and intermediate positions
 - Ensure that all four casters lock when the brake pedal is engaged
 - Run through each function on the foot board control panel to ensure that each function is working properly.
 - Run through each function on both head end siderails to ensure that each is working properly.

If any problems are found during bed set-up, contact Stryker Customer Service at 800-327-0770.

Damaged Merchandise

ICC Regulations require that claims for damaged merchandise must be made with the carrier within fifteen (15) days of receipt of merchandise. **DO NOT ACCEPT DAMAGED SHIPMENTS UNLESS SUCH DAMAGE IS NOTED ON THE DELIVERY RECEIPT AT THE TIME OF RECEIPT.** Stryker Customer Service must be notified immediately. Stryker will aid the customer in filing a freight claim with the appropriate carrier for damages incurred. Claim will be limited in amount to the actual replacement cost. In the event that this information is not received by Stryker within the fifteen (15) day period following the delivery of the merchandise, or the damage was not noted on the delivery receipt at the time of receipt, the customer will be responsible for payment of the original invoice in full.

Claims for any short shipment must be made within thirty (30) days of invoice.

Bed Symbols



Warning, Refer to Service/Maintenance Manual



Alternating Current



Type B Equipment: equipment providing a particular degree of protection against electric shock, particularly regarding allowable leakage current and reliability of the protective earth connection.

Class 1 Equipment: equipment in which protection against electric shock does not rely on BASIC INSULATION only, but which includes an additional safety precaution in that means are provided for the connection of the EQUIPMENT to the protective earth conductor in the fixed wiring of the installation in such a way that ACCESSIBLE METAL PARTS cannot become live in the event of a failure of the BASIC INSULATION.

IPX4: Protection from liquid splash



Dangerous Voltage Symbol



Protective Earth Terminal




Potential Equalization Symbol



Medical Equipment Classified by Underwriters Laboratories Inc. with Respect to Electric Shock, Fire, Mechanical and Other Specified Hazards Only in Accordance with UL 2601-1 and CAN/CSA C22.2 No. 601.1

Bed Symbols

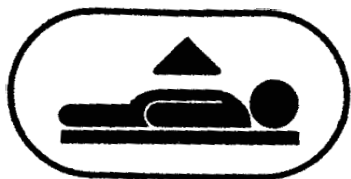

Lift Pedal

Pump pedal to raise bed. Hold pedal down to activate motorized pump operation (when the bed power cord is plugged into the wall outlet).



Head Down Pedal


Depress pedal to lower head end.

Note: Depress both Head and Foot end pedals at the same time to lower litter.



Descent Pedal


Depress center of pedal to lower both ends of the litter together.

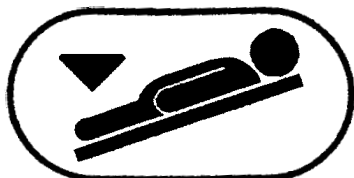

Foot Down Pedal

Depress pedal to lower foot end.


Note: Depress both Head and Foot end pedals at the same time to lower litter.

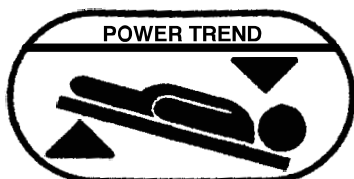

Power Trendelenburg Pedal


Depress pedal to lower head end and raise foot end simultaneously (when the bed power cord is plugged into the wall outlet).

Bed Symbols

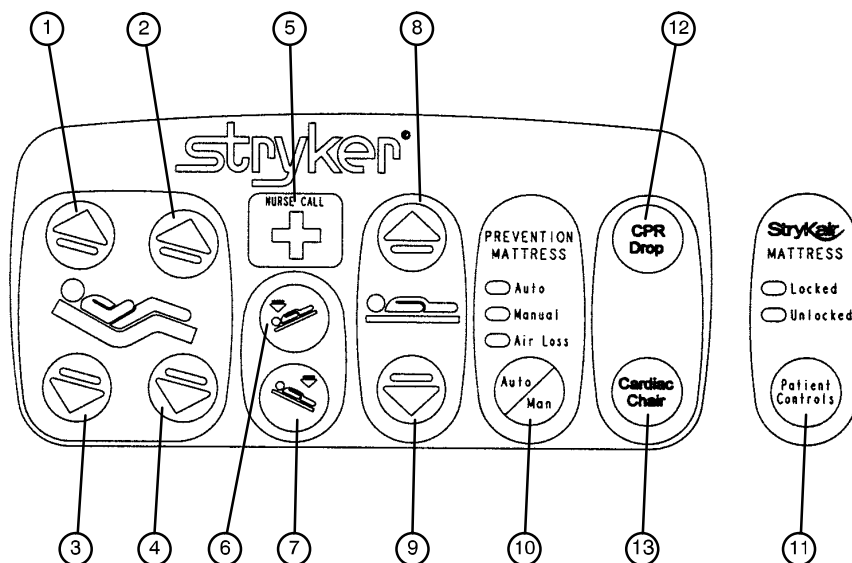

1. Press to raise back section.
2. Press to raise knee section.
3. Press to lower back section.
4. Press to lower knee section.
5. Press to activate nurse call.
6. Press to lower the head end of the bed (Trendelenburg).
7. Press to lower the foot end of the bed (Reverse Trendelenburg).
8. Press to raise the litter.
9. Press to lower the litter.
10. Press to activate the automatic or manual operation of the Dynamic Mattress System™.
11. Press to lock out patient control of the StryKair™ Mattress.
12. Press to activate emergency CPR positioning.
13. Press to activate emergency Cardiac Chair positioning.



Bed Symbols

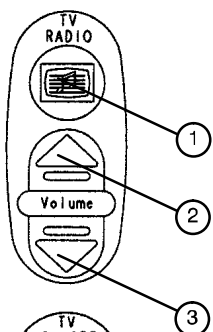

1. Press to turn on the TV or radio. Press again to change TV channels and to turn off the TV.

2. Press to increase the TV or radio volume.

3. Press to decrease the TV or radio volume.

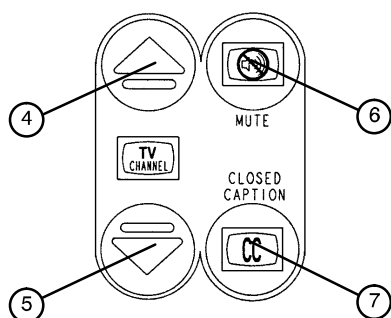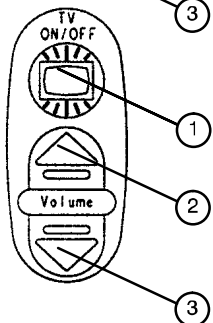


4. Press to change the TV channel up.

5. Press to change the TV channel down.


6. Press to mute the TV sound. Press again to turn the sound back on.

7. Press to display TV closed captioning.

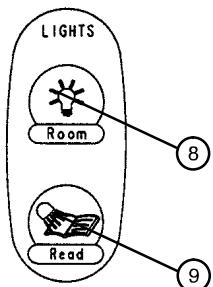


8. Press to turn on the room lights. Press again to turn off.

9. Press to turn on the reading light. Press again to turn off.

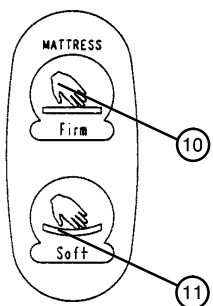


10. Press to increase the firmness of the mattress.

11. Press to decrease the firmness of the mattress.

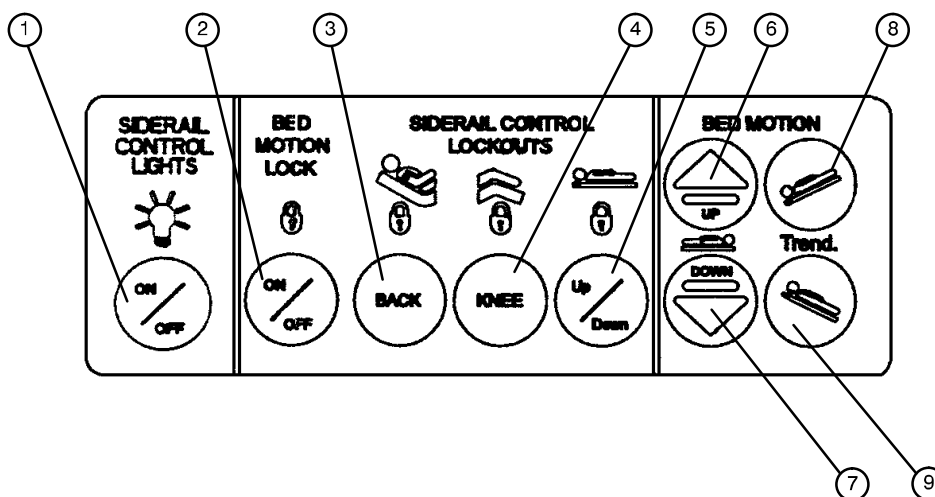


1. Press repeatedly for low, medium and high settings for the siderail control lights. Continue to press this switch to turn off the siderail control lights and the nurse call indicator light.
2. Press to lock out all bed motion controls on the siderails. Press again to unlock.
3. Press to lock out Back motion control on the siderails. Press again to unlock.
4. Press to lock out Knee motion control on the siderails. Press again to unlock.
5. Press to lock out bed up/down motion controls on the siderails. Press again to unlock.
6. Press to raise bed.
7. Press to lower bed.
8. Press to lower head end of bed (Trendelenburg).
9. Press to lower foot end of bed (Reverse Trendelenburg).

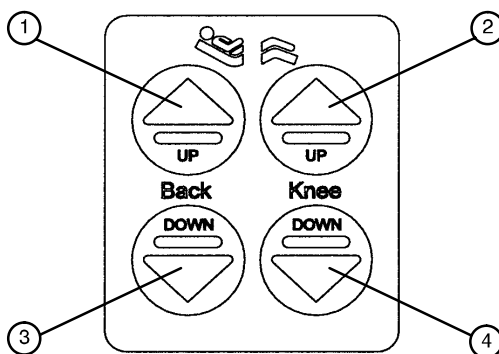


1. Press to raise back section.
2. Press to raise knee section.
3. Press to lower back section.
4. Press to lower knee section.

Bed Symbols

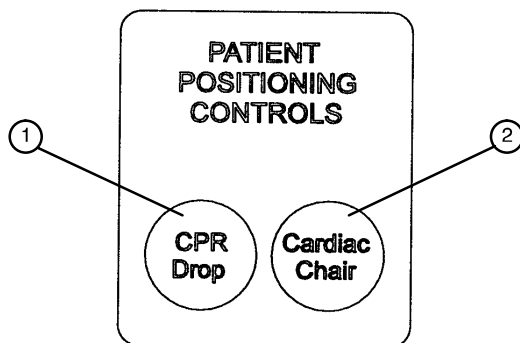


1. Press to activate the emergency CPR drop function. The bed will level from Trendelenburg/reverse Trendelenburg, the Fowler will lower to flat, the Knee will lower to flat and the litter will lower to full down.
2. Press to activate the emergency Cardiac Chair function. The Knee will raise, the Fowler will raise or lower to 51° and the bed will tilt to -10° reverse Trendelenburg (foot end down). Release the button to stop bed movement: hold the button until movement stops to complete the function.

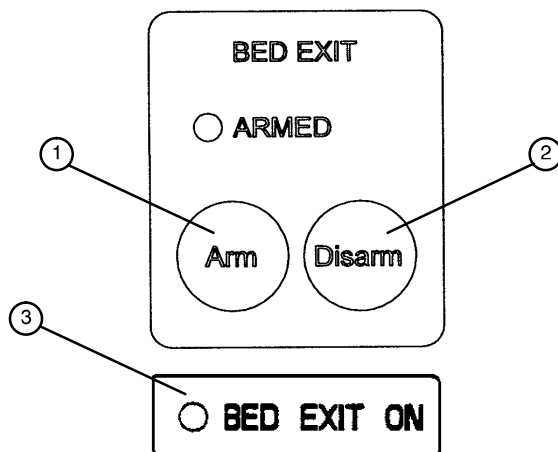


1. Press to arm the Bed Exit function.
2. Press to disarm the Bed Exit function.
3. "BED EXIT ON" LED – will light when the BED EXIT function is armed.

Bed Symbols

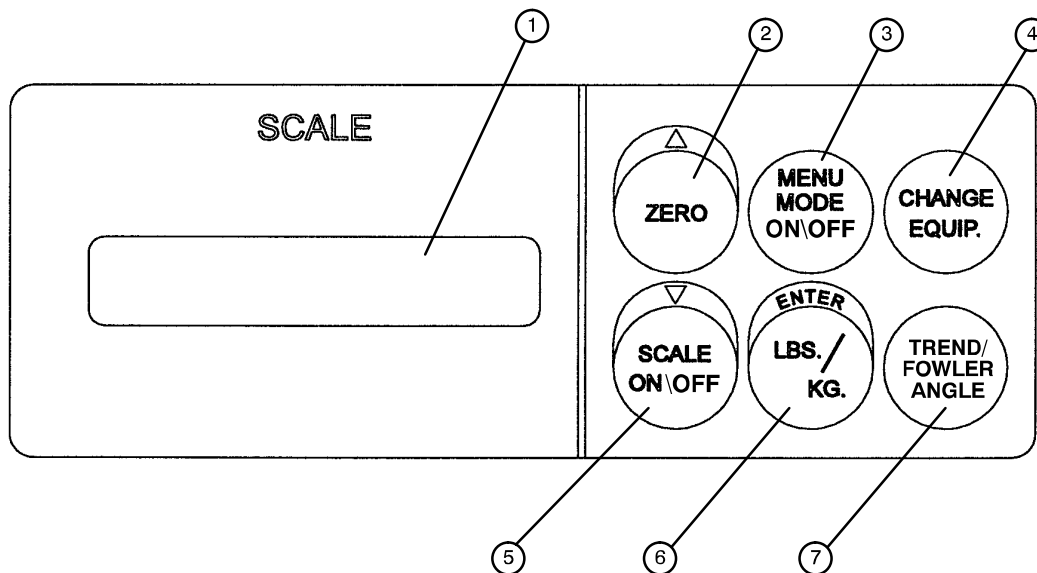

1. LCD – displays patient weight. Trendelenburg angle is displayed when the scale is not active.
2. Press to zero bed. Also press to scroll while Menu Mode is active.
3. Press to enter and exit the Menu Mode.
4. Press when adding or removing equipment to the bed.
5. Press to turn weigh system on and off. Also press to scroll while Menu Mode is active.
6. Press to change weight from pounds to kilograms or back. Also press while using the Menu Mode.
7. Press to display the Trendelenburg or Fowler angle of the bed.

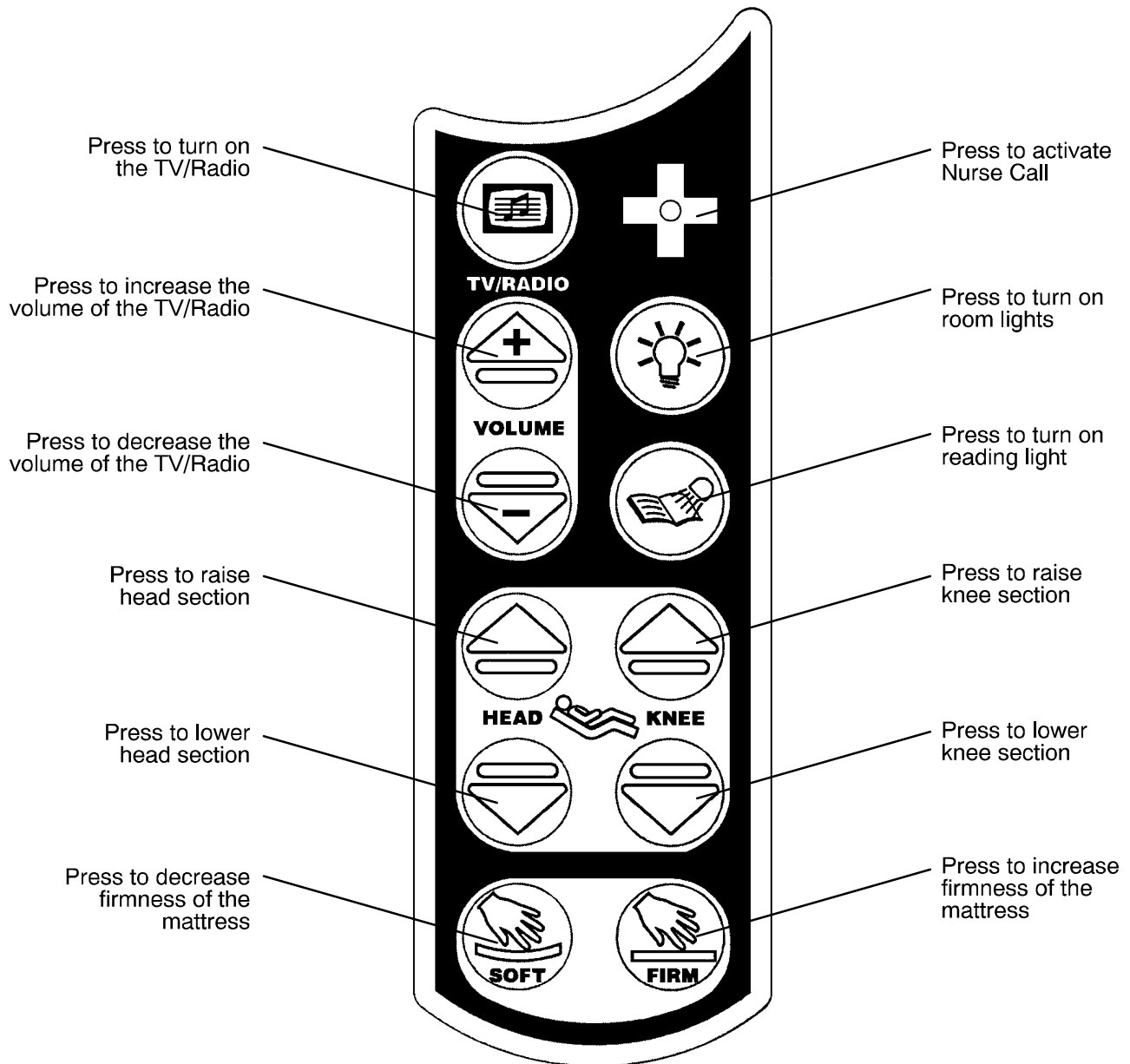

GENERAL INFORMATION

This section contains cleaning instructions and a checklist to assist with the routine preventive maintenance and cleaning of your equipment.

In the text, the words “right” and “left” refer to the right and left sides of a patient lying face up on the bed.

PREVENTIVE MAINTENANCE CONTENTS

Cleaning 2-2

Preventive Maintenance Checklist 2-3

General Information 2-4

Cleaning

Hand wash all surfaces of the bed with warm water and mild detergent. Dry thoroughly. **DO NOT STEAM CLEAN, PRESSURE WASH, HOSE OFF OR ULTRASONICALLY CLEAN.** Using these methods of cleaning is **not** recommended and may void this product's warranty.

Clean Velcro **AFTER EACH USE**. Saturate Velcro with disinfectant and allow disinfectant to evaporate. (Appropriate disinfectant for nylon Velcro should be determined by the hospital.)

In general, when used in those concentrations recommended by the manufacturer, either phenolic type or quaternary type disinfectants can be used with Staph–Chek fabrics. Iodophor type disinfectants are not recommended for use on Staph–Chek fabrics because staining may result. The following products have been tested by the Herculite Laboratory and have been found not to have a harmful effect on Staph–Chek fabrics **WHEN USED IN ACCORDANCE WITH MANUFACTURERS RECOMMENDED DILUTION.***

TRADE NAME	DISINFECTANT TYPE	MANUFACTURER	*MANUFACTURER'S RECOMMENDED DILUTION
A33	Quaternary	Airwick (Professional Products Division)	2 ounces/gallon
A33 (dry)	Quaternary	Airwick (Professional Products Division)	1/2 ounce/gallon
Beaucoup	Phenolic	Huntington Laboratories	1 ounce/gallon
Blue Chip	Quaternary	S.C. Johnson	2 ounces/gallon
Elimstaph	Quaternary	Walter G. Legge	1 ounce/gallon
Franklin Phenomysan F2500	Phenolic	Purex Corporation	1 1/4 ounce/gallon
Franklin Sentinel	Quaternary	Purex Corporation	2 ounces/gallon
Galahad	Phenolic	Puritan Churchill Chemical Company	1 ounce/gallon
Hi–Tor	Quaternary	Huntington Laboratories	1/2 ounce/gallon
LPH	Phenolic	Vestal Laboratories	1/2 ounce/gallon
Matar	Phenolic	Huntington Laboratories	1/2 ounce/gallon
Omega	Quaternary	Airwick (Professional Products Division)	1/2 ounce/gallon
Quanto	Quaternary	Huntington Laboratories	1 ounce/gallon
Sanikleen	Quaternary	West Chemical Products	2 ounces/ gallon
Sanimaster II	Quaternary	Service Master	1 ounce/gallon
Vesphene	Phenolic	Vestal Laboratories	1 1/4 ounce/ gallon

Quaternary Germicidal Disinfectants, used as directed, and/or Chlorine Bleach products, typically 5.25% Sodium Hypochlorite in **dilutions ranging between 1 part bleach to 100 parts water, and 2 parts bleach to 100 parts water are not considered mild detergents. These products are corrosive in nature and may cause damage to your stretcher if used improperly.** If these types of products are used to clean Stryker patient handling equipment, measures must be taken to insure the stretchers are rinsed with clean water and thoroughly dried following cleaning. Failure to properly rinse and dry the stretchers will leave a corrosive residue on the surface of the stretcher, possibly causing premature corrosion of critical components.

NOTE

Failure to follow the above directions when using these types of cleaners may void this product's warranty.

REMOVAL OF IODINE COMPOUNDS

This solution may be used to remove iodine stains from mattress cover and foam footrest pad surfaces.

1. Use a solution of 1–2 tablespoons Sodium Thiosulfate in a pint of warm water to clean the stained area. Clean as soon as possible after staining occurs. If stains are not immediately removed, allow solution to soak or stand on the surface.
2. Rinse surfaces which have been exposed to the solution in clear water before returning bed to service.

Preventive Maintenance Checklist

- _____ All fasteners secure
- _____ Engage brake pedal and push on the bed to ensure all casters lock securely
- _____ Optional locking steer caster or Fifth Wheel engages and disengages properly
- _____ Siderails move, latch and stow properly
- _____ All functions on siderails working properly (including LED's)
- _____ CPR release working properly
- _____ Foot prop intact and working properly
- _____ I.V. pole working properly
- _____ Optional Foley bag hooks intact
- _____ Optional chart rack intact and working properly
- _____ Optional CPR board not cracked or damaged and stores properly
- _____ No cracks or splits in head and foot boards
- _____ All functions on footboard working properly (including LED's)
- _____ No rips or cracks in mattress cover
- _____ No hydraulic hoses worn, kinked, or leaking
- _____ No leaks at hydraulic connections
- _____ Power cord not frayed
- _____ No cables worn or pinched
- _____ All electrical connections tight
- _____ All grounds secure to the frame
- _____ Ground impedance not more than 100 milliohms
- _____ Current leakage not more than 100 microamps

Bed Serial No. _____

Completed By: _____ Date: _____

NOTE


Preventative maintenance should be performed at a minimum of annually. A preventative maintenance program should be established for all Stryker Medical equipment. Preventative maintenance may need to be performed more frequently based on the usage level of the product.

General Information

NOTE

To prevent a low battery condition when the bed is not plugged in, position the cord out switch at the head end of the bed to the off position. The switch is identified by the label shown below. If the switch is not positioned as shown below and the bed power cord and pendant cord are unplugged, the life of the back-up battery will be significantly reduced.

If the power light (located on the foot board) is flashing, the Nurse Call battery needs to be replaced. The battery is located on the patient's left side at the head end of the bed. No tools are required to replace the battery. Unplug the bed power cord from the wall socket and replace the battery.

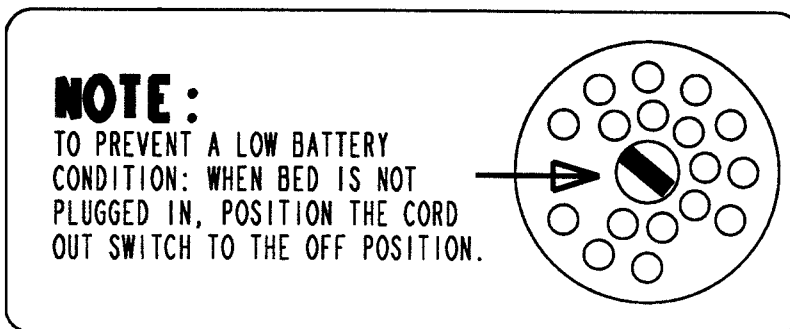

GENERAL INFORMATION

This section contains troubleshooting charts to assist with the diagnosis of a problem with your equipment. In the text, the words “right” and “left” refer to the right and left sides of a patient lying face up on the bed.

TROUBLESHOOTING CONTENTS

Troubleshooting Guide 3-2

Optional Dynamic Mattress System Troubleshooting Guide 3-3

Troubleshooting Guide

DEFINITIONS:

DMM = Digital Multi-Meter
 PCB = Printed Circuit Board
 CPU = Central Processing Unit

NOTE

See page 4–2 through page 4–8 for an outline of bed PCB's and voltage test points.

PROBLEM/FAILURE	RECOMMENDED ACTION
No power to bed	<p>A. Check circuit breaker on bed.</p> <p>B. Check for 120 VAC power at J1 on power supply. See page 4–4 for power supply voltage test points.</p> <p>C. Check for DC voltages on J2 (Pins 1,2,3 & 6) on power supply. See page 4–4 for power supply voltage test points.</p>
No bed down motion.	<p>A. Enter diagnostics, (see page 7–2) and press bed down. If motion is present, re-burn lift potentiometers. Monitor Pin 3 and Pin 2 of HDR7 and HDR12 on the CPU PCB using DMM. Verify voltage changes on Pin 3 with changes in lift motion. See page 7–8 for voltage parameters for low and high limits.</p> <p>B. If no down motion in diagnostic, check for 120 VAC power on HDR33 and HDR34, Pin 1 and Pin 3, of the CPU.</p> <p>C. Check for 1.1–1.5 VDC signal on O6 and O8 Pin 1 and HDR2 Pin 5 of the CPU PCB.</p> <p>D. Check for motion interrupt jumper on HDR3.</p>
No bed up motion.	<p>A. Check 120 VAC power on HDR33, Pin 1 and Pin 6, of the CPU board.</p> <p>B. Check for 1.1–1.5 VDC signal on O5 Pin 1 and HDR2 Pin 5 of the CPU PCB.</p>
No Gatch down motion.	<p>A. Check for 120 VAC power on HDR30 Pin 1 and Pin 3 of the CPU board.</p> <p>B. Check for 1.1–1.5 VDC signal on O3 Pin 1 and HDR2 Pin 5 of the CPU PCB.</p>
No Gatch up motion.	<p>A. Check for 120 VAC on HDR30, Pin 1 and Pin 2 of the CPU board.</p> <p>B. Check for 1.1–1.5 VDC on O1 Pin 1 and HDR2 Pin 5 of the CPU PCB.</p>
No Fowler down motion.	<p>A. Check for 120 VAC power on HDR29 Pin 3 and Pin 1 of the CPU board.</p> <p>B. Check for 1.1–1.5 VDC signal on O4 Pin 1 and HDR2 Pin 5 of the CPU PCB.</p>
No Fowler up motion.	<p>A. Check for 120 VAC on HDR29, Pin 1 and Pin 3 of the power supply.</p> <p>B. Check for 1.1–1.5 VDC on O2 Pin 1 and HDR2 Pin 5 of the CPU PCB.</p>

Optional Dynamic Mattress System Troubleshooting Guide

PROBLEM/SYMPTOM	SOLUTION(S)
AIR LOSS LED on.	<p>A. Check air connections between bladder and control unit.</p> <p>B. Check for leak in bladder.</p> <p>C. Unplug transformer from wall socket and plug back in.</p> <p>D. Replace bladder (page 10–3).</p> <p>E. Replace control unit (page 10–5).</p>
Auto LED does not light when plugged in.	<p>A. Plug into another wall socket.</p> <p>B. Open control unit and check for 12 VDC at power supply connector.</p> <p>C. Replace power supply assembly</p> <p>D. Replace control unit (page 10–5).</p>
Cover stained, upper foam layer stained.	<p>A. Check cleaning solution and protocol.</p> <p>B. Replace outer cover (page 10–2).</p> <p>C. Replace upper foam topper.</p>

GENERAL INFORMATION


This section contains circuit board layouts and other information on the electrical system of the bed.

ELECTRICAL SYSTEM INFORMATION CONTENTS

CPU Board 4-2, 4-3
Power Supply 4-4
Optional DMS Control Board 4-5
Optional StryKair Power Supply 4-6
Optional Smart TV Circuit Board 4-7, 4-8
Bed Communications Tester 4-9
Head Wall Output Configuration 4-10

CPU Board

- CPU KIT (SCALE ONLY) - 2035-700-001
- CPU KIT (BED EXIT ONLY) - 2035-700-002
- CPU KIT (SCALE & BED EXIT) - 2035-700-003

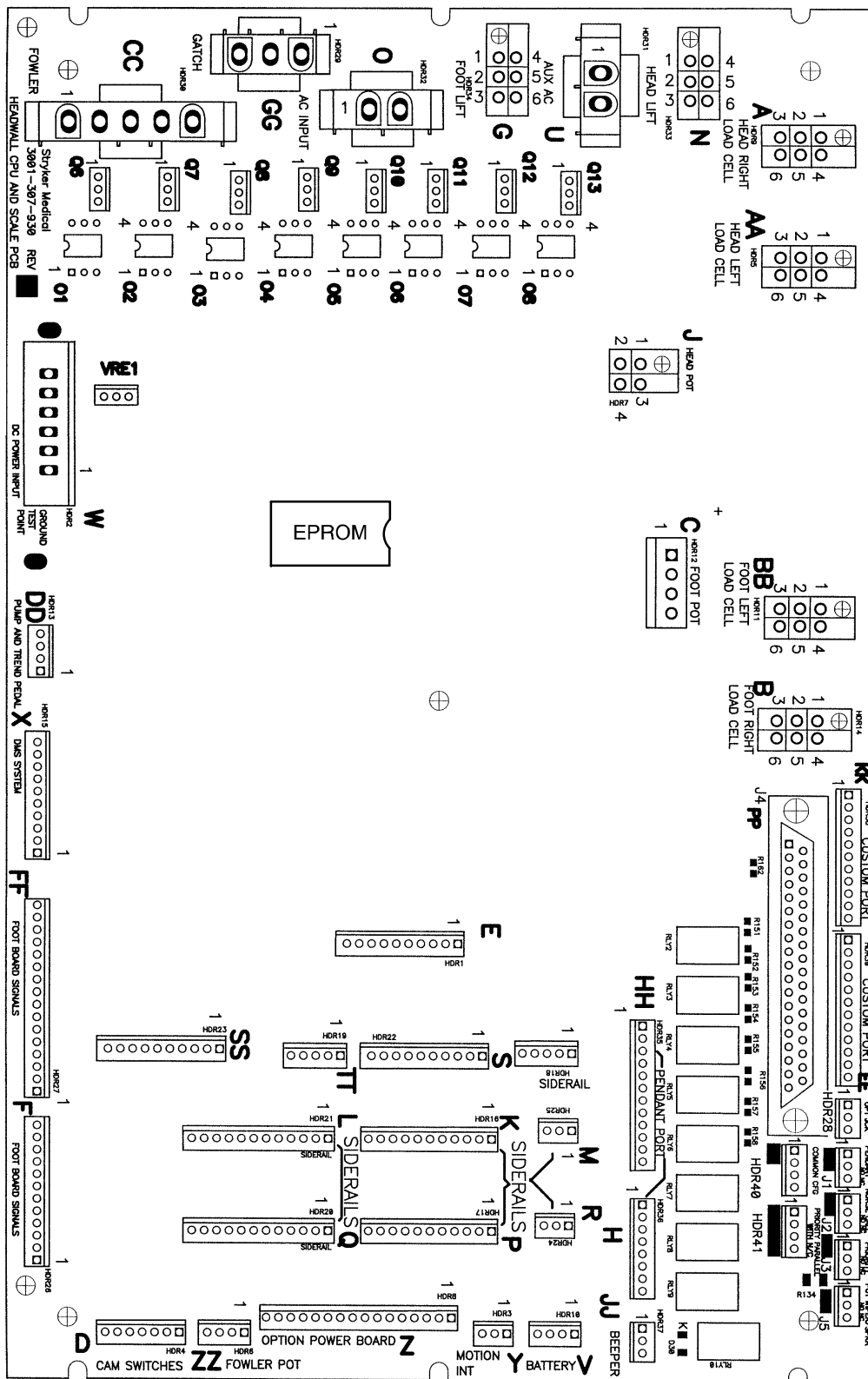

CPU Board (Continued)

CPU KIT (SCALE ONLY) – 2035-700-001


CPU KIT (BED EXIT ONLY) – 2035-700-002

CPU KIT (SCALE & BED EXIT) – 2035-700-003

CONNECTOR LOCATION	CABLE LOCATION	VOLTAGE	POSITIVE LEAD	NEGATIVE LEAD	DESCRIPTION
HDR 2	W	+12 VDC	Pin 1	Pin 4 or 5	Relays & Siderails Light Voltage
HDR 2	W	+5 VDC	Pin 2 & 3	Pin 4 or 5	+5 VDC from Power Supply
HDR 2	W	-12 VDC	Pin 6	Pin 4 or 5	Relays & Siderails Light Voltage
HDR 6	ZZ	+5 VDC	Pin 1	Pin 4	+5 VDC for Fowler Pot
HDR 6	ZZ	0 – 5 VDC	Pin 3	Pin 4	Fowler Pot Wiper
HDR 7	J	0 – 5 VDC	Pin 3	Pin 2	Head Lift Pot Wiper
HDR 7	J	+5 VDC	Pin 4	Pin 2	+5 VDC for Head Lift Pot
HDR 12	C	+5 VDC	Pin 1	Pin 2	+5 VDC for Foot Lift Pot
HDR 12	C	0 – 5 VDC	Pin 3	Pin 2	Foot Lift Pot Wiper
HDR 29	CC	0 VAC w/o Switch 110 VAC w/Switch	Neutral Pin 1	Pin 2	Gatch Up
HDR 29	CC	0 VAC w/o Switch 110 VAC w/Switch	Neutral Pin 1	Pin 3	Gatch Down
HDR 30	GG	0 VAC w/o Switch 160 VAC w/ Switch	Neutral Pin 3	Pin 1	Fowler Up
HDR 30	GG	0 VAC w/o Switch 120 VAC w/ Switch	Neutral Pin 3	Pin 2	Fowler Down
HDR 32	O	110 VAC	Pin 1	Pin 2	Line Voltage to Bed
HDR 33	N	0 VAC w/o Switch 120 VAC w/ Switch	Neutral Pin 1	Pin 3	Head Lift Down
HDR 33	N	0 VAC w/o Switch 120 VAC w/ Switch	Neutral Pin 1	Pin 6	Bed Up (Pump Motor)
HDR 34	G	0 VAC w/o Switch 120 VAC w/ Switch	Neutral Pin 1 or 4	Pin 3	Foot Lift Down

Power Supply


POWER SUPPLY – P/N 59-157

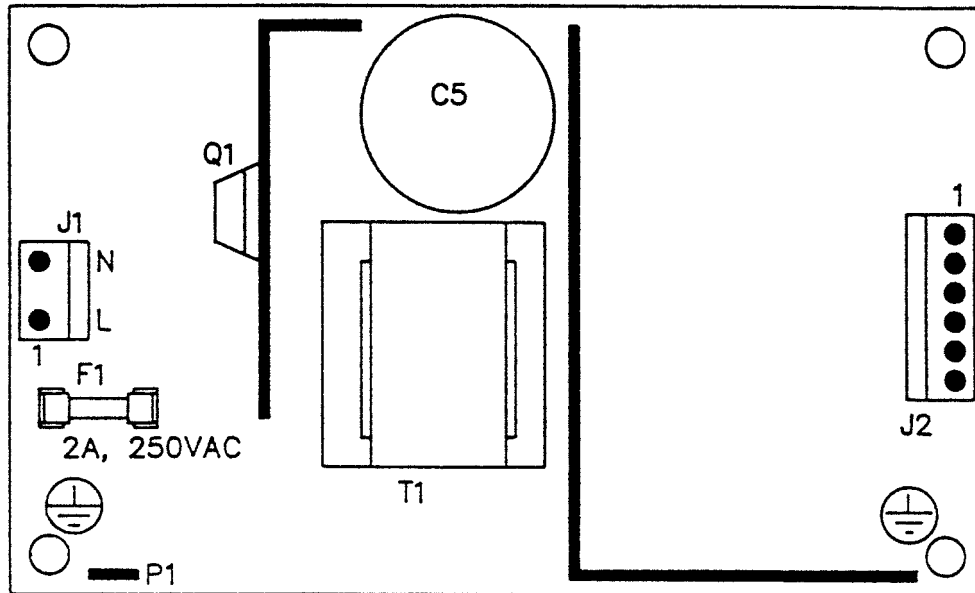

CONNECTOR LOCATION	VOLTAGE	POSITIVE LEAD	NEGATIVE LEAD
J1	110V	Pin 1	Pin 2
J2	12V	Pin 1	Pin 4 or 5
J2	5V	Pin 2	Pin 4 or 5
J2	5V	Pin 3	Pin 4 or 5
J2	GND	Pin 4	Pin 4 or 5
J2	GND	Pin 5	Pin 4 or 5
J2	-12V	Pin 6	Pin 4 or 5

DMS Control Board


OPTIONAL INTEGRATED DMS CONTROL BOARD – P/N 2500-402-900

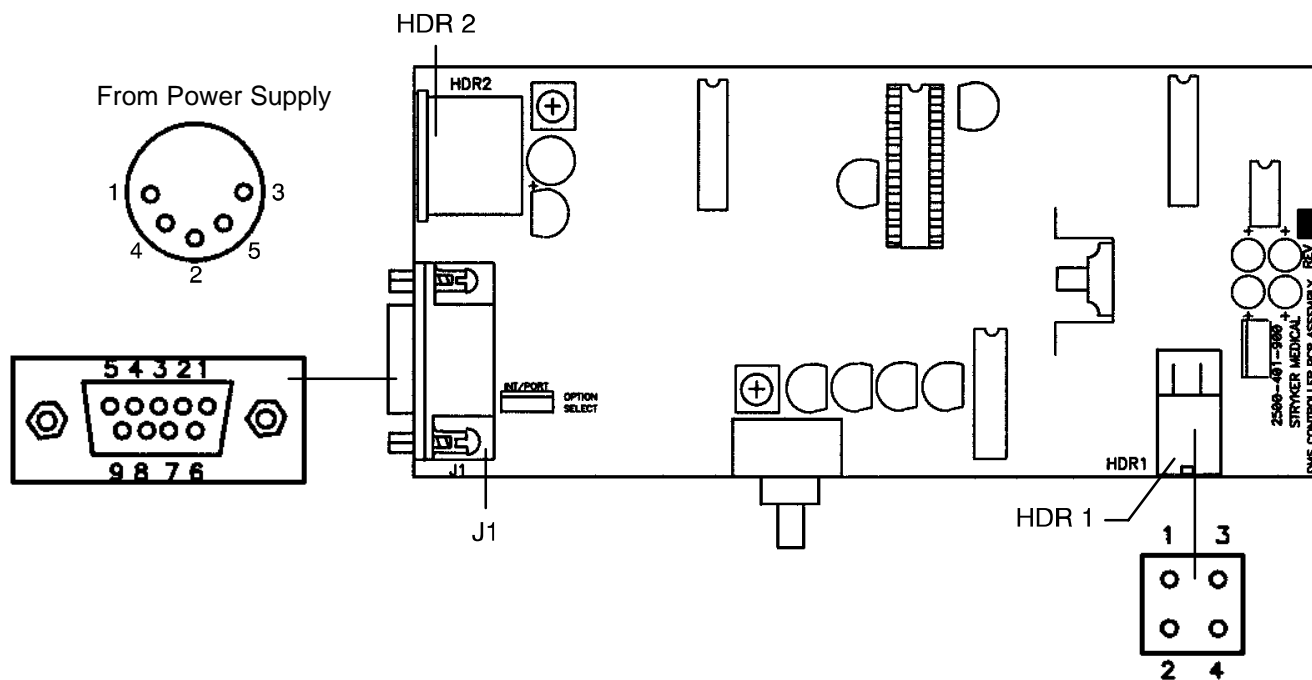

CONNECTOR LOCATION	VOLTAGE	POSITIVE LEAD	NEGATIVE LEAD	DESCRIPTION
PS	+12 – +14 VDC	Pin 1	Pin 3	Power Supply
PS	+12 – +14 VDC	Pin 4	Pin 3	Power Supply
HDR 1	+12 – +14 VDC	Pin 4	Pin 3	Air Compressor
J1	+5 VDC	Pin 2	Pin 3	Hand Pendant
J1	Digital (Ref.)	Pin 6	Pin 3	Manual LED
J1	Digital (Ref.)	Pin 7	Pin 3	Auto LED
J1	Digital (Ref.)	Pin 8	Pin 3	Error LED
J1	+12 – +14 VDC	Pin 9	Pin 3	Power Supply

StryKair Power Supply


OPTIONAL STRYKAIR POWER SUPPLY – P/N 2600-300-900

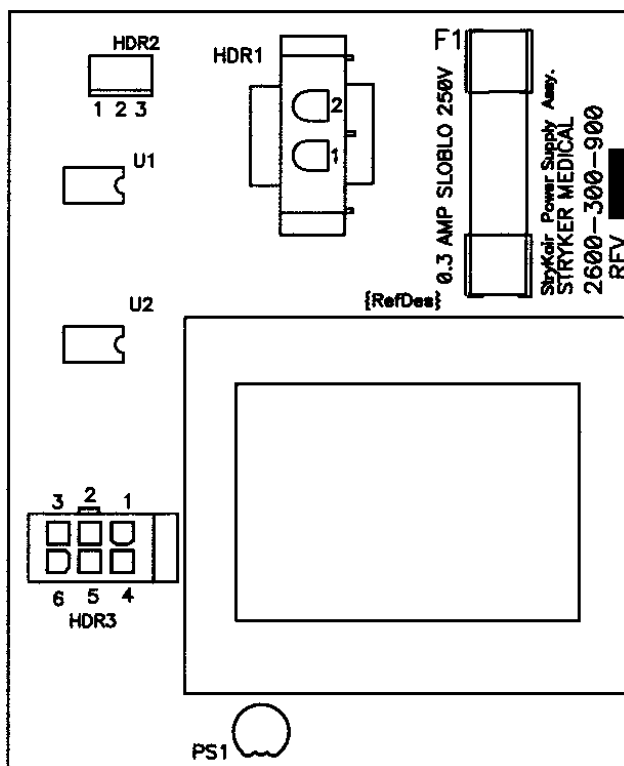

CONNECTOR LOCATION	VOLTAGE	POSITIVE LEAD	NEGATIVE LEAD	DESCRIPTION
HDR 1	120 VAC	Pin 1	Pin 2	AC power to board
HDR 3	12-16 VAC	Pin 1	Pin 2	AC power to mattress
HDR 3	+ 5 VDC	Pin 4	Pin 3	DC power returned to board
HDR 2	+ 5 VDC	Pin 1	Pin 3	S/R switch voltage with firm switch pressed
HDR 2	+ 5 VDC	Pin 2	Pin 3	S/R switch voltage with soft switch pressed

Smart TV Circuit Board


OPTIONAL SMART TV CIRCUIT BOARD – P/N 3001–330–950

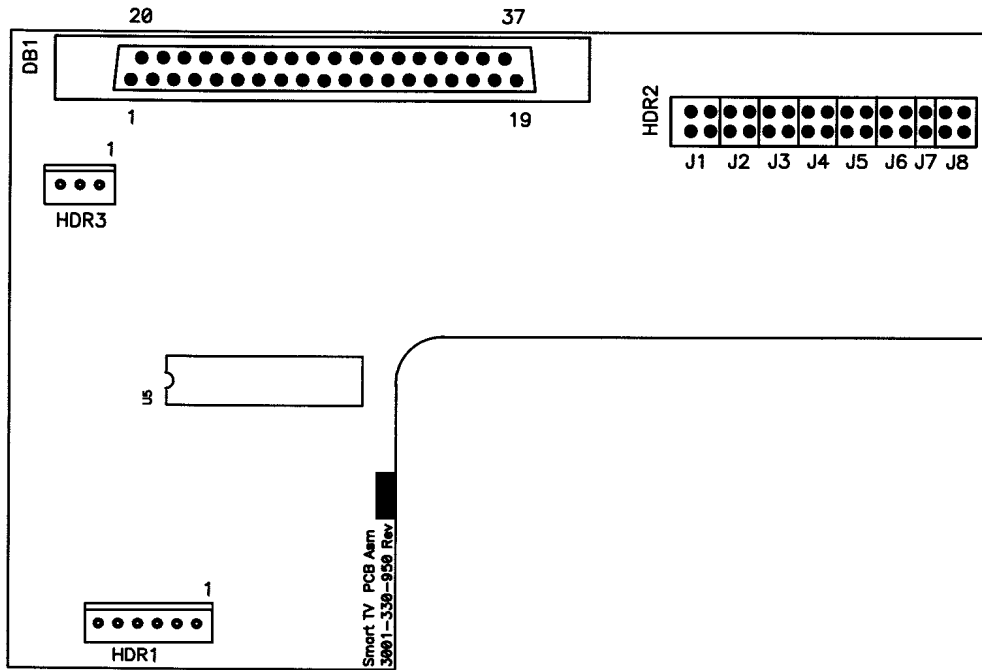

CONNECTOR LOCATION	VOLTAGE	POSITIVE LEAD	NEGATIVE LEAD	DESCRIPTION
HDR 1	5 VDC	Pin 2	Pin 1	Regulated 5 VDC Power to the board
HDR 1	Digital Control	Pin 3–5	Pin 1	Serial control lines
HDR 1	5 VDC	Pin 6	Pin 1	5 VDC for option relay
HDR 3	+5 or +12 VDC	2	Pin 1	Power/control line from the TV Note: This header provides TV control to a non–Stryker pendant
DB1	+5 or +12 VDC	Pin 34	Pin 33	Power/Control line from the TV Note: If this polarity is reversed, place the shunts of J8 in the alternate position

Smart TV Circuit Board


OPTIONAL SMART TV CIRCUIT BOARD – P/N 3001-330-970

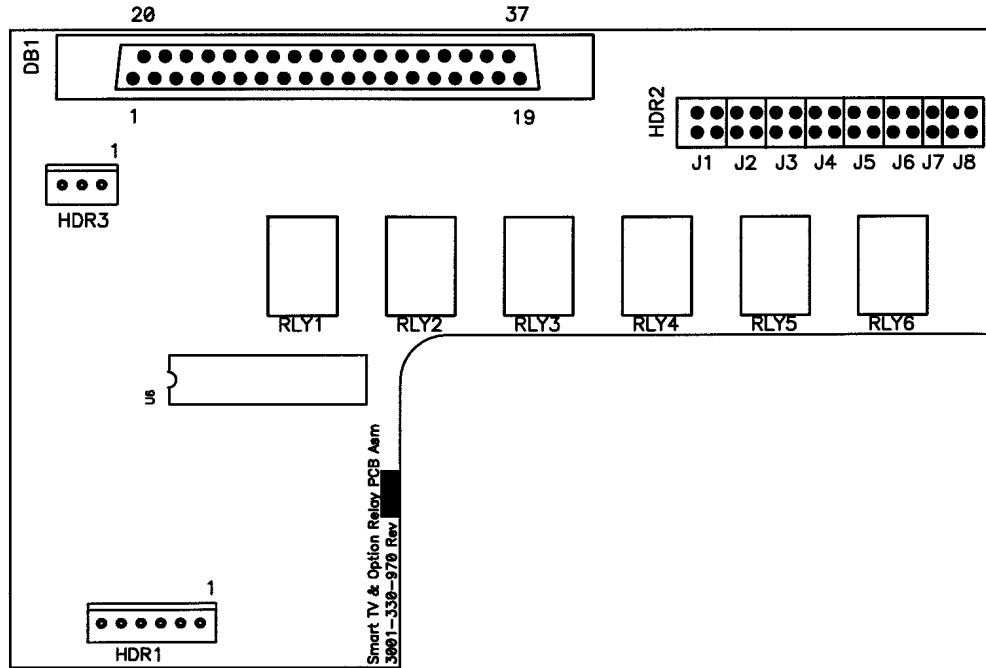

CONNECTOR LOCATION	VOLTAGE	POSITIVE LEAD	NEGATIVE LEAD	DESCRIPTION
HDR 1	5 VDC	Pin 2	Pin 1	Regulated 5 VDC Power to the board
HDR 1	Digital Control	Pin 3-5	Pin 1	Serial control lines
HDR 1	5 VDC	Pin 6	Pin 1	5 VDC for option relay
HDR 3	+5 or +12 VDC	2	Pin 1	Power/control line from the TV Note: This header provides TV control to a non-Stryker pendant
DB1	+5 or +12 VDC	Pin 34	Pin 33	Power/Control line from the TV Note: If this polarity is reversed, place the shunts of J8 in the alternate position

Bed Communications Tester


OPTIONAL BED COMMUNICATIONS TESTER – P/N 3001-303-165

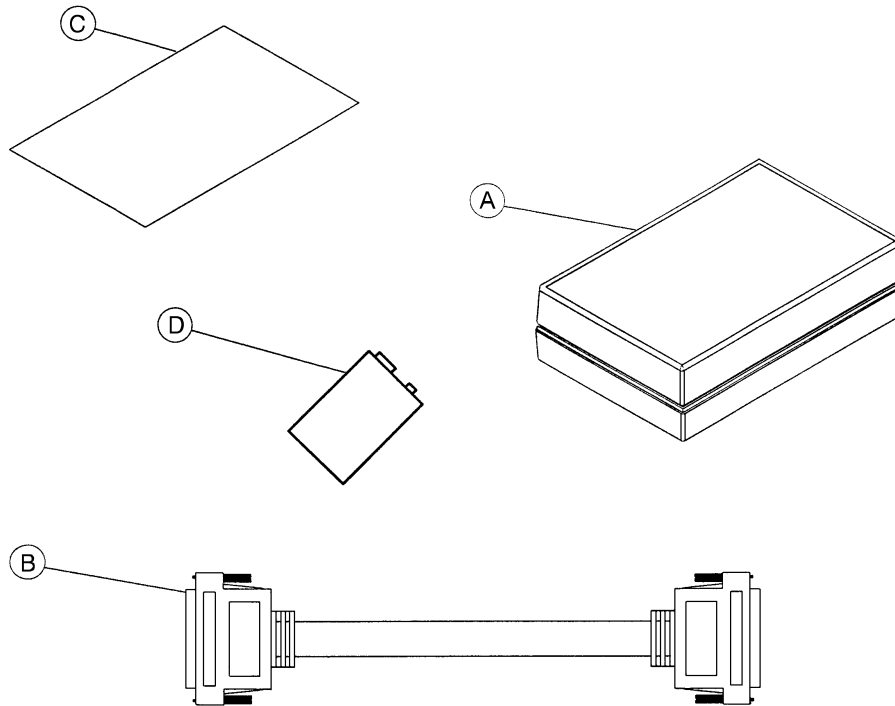

Item	Part No.	Part Name	Qty.
A	3001-303-160	BCT Unit	1
B	3001-303-825	37-Pin Cable	1
C	3001-303-162	Instructions	1
D	3000-303-871	9V Battery	1


Head Wall Output Configuration

37-PIN CONNECTOR

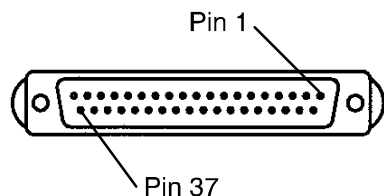

Pin 1	Option 2 Common
Pin 2	Read Light
Pin 3	Room Light
Pin 4	Speaker High
Pin 5	Pot Wiper
Pin 6	Radio Common
Pin 7	Nurse Call Interlock
Pin 8	Audio Transfer –
Pin 9	Audio Transfer +
Pin 10	Interlock +
Pin 11	Interlock –
Pin 12	Spare
Pin 13	Options 3 Common
Pin 14	Pot Low Common
Pin 15	Pot High Common
Pin 16	Nurse Answer Light +
Pin 17	Option 1 NO/NC
Pin 18	Option 1 Common
Pin 19	Nurse Call Light +
Pin 20	Option 2 NO/NC
Pin 21	Option 3 NO/NC
Pin 22	Option 3A NO/NC
Pin 23	Option 2A Common
Pin 24	Option 2A NO/NC
Pin 25	Nurse Call +
Pin 26	Nurse Call NO/NC
Pin 27	Room/Read Light Common
Pin 28	Nurse Call Light –
Pin 29	Nurse Answer Light –
Pin 30	Priority NO/NC
Pin 31	Priority Common
Pin 32	Option 3A Common
Pin 33	TV –
Pin 34	TV +
Pin 35	Speaker Low Common
Pin 36	Audio Shield
Pin 37	Radio NO/NC

STRYKER PENDANT PORT

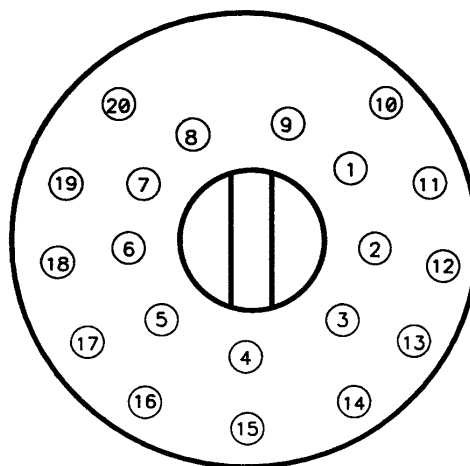

1	Scan Line
2	Audio (–)
3	Nurse Call (+)
4	+5 VDC
5	Scan Line
6	Scan Line
7	Nurse Call (–)
8	TV Channel Up
9	Backlight
10	Audio (+)
11	Gatch Up/Fowler In/Foot Up/DMS Firm
12	Gatch Down/Fowler Out/Foot Out/DMS Soft
13	Fowler Up/Trend In
14	Fowler Down/Trend Out
15	Audio Shield
16	Not Used – Socket Filled
17	Bed Up
18	Ground
19	Read Light/Bed Down
20	Room Light

ELECTRICAL COMPONENTS

FOOT BOARD KEYBOARD (S/R LIGHTS, LOCKOUTS, ETC.)	3001-500-900
FOOT BOARD SCALE DISPLAY	3001-507-900
FOOT BOARD SCALE KEYBOARD	3001-507-910
FOOT BOARD BED EXIT KEYBOARD	3001-508-900
CPU KIT (SCALE ONLY)	2035-700-1
CPU KIT (BED EXIT ONLY)	2035-700-2
CPU KIT (SCALE & BED EXIT)	2035-700-3
POWER SUPPLY	59-157
SMART TV CIRCUIT BOARD	3001-330-950
SMART TV CIRCUIT BOARD	3001-330-970

SIDERAIL BOARDS

DMS BOARD	3001-402-900
INSIDE BOARD	3001-400-930
OUTSIDE BOARD	3001-400-910
SMART TV BOARD, RIGHT	5000-400-920
SMART TV BOARD, LEFT	5000-400-930
SPEAKER W/CABLE	3000-403-831

OTHER COMPONENTS

CAPACITOR, FOWLER & GATCH	3000-300-453
CAPACITOR, FOWLER & GATCH, 230V	3221-300-401
CAPACITOR, LIFT	59-140
CAPACITOR, LIFT, 230V	59-156
CASTER, 8"	2025-10-25
CASTER, STEER, 8"	2025-10-24
COIL CORD, LIFT POWER	2025-201-802
COIL CORD, LIFT SENSOR	2035-1-801
COMMUNICATIONS TESTER	3001-303-165
LOAD CELL	3001-307-57
MOTOR, FOWLER & GATCH W/CLUTCH	3001-300-705
MOTOR, FOWLER & GATCH W/CLUTCH, 230V	3221-300-705

OTHER COMPONENTS (CONTINUED)

PAINT, TOUCH-UP, OPAL, BOTTLE W/BRUSH	7000-1-321
PAINT, TOUCH-UP, OPAL, SPRAY CAN	7000-1-318
POTENTIOMETER, FOOT & HEAD LIFT	2025-1-807
POTENTIOMETER, FOWLER	2025-232-803
POWER CORD	39-254
SINGLE TUBE OF GREASE	3000-200-700

GENERAL INFORMATION

This section contains tool lists and step-by-step procedures to assist with the maintenance and servicing of the base portion of your equipment.

In the text, the words “right” and “left” refer to the right and left sides of a patient lying face up on the bed.

BASE MAINTENANCE CONTENTS

Static Discharge Precautions 6-2

Checking Hydraulic Fluid Level 6-3

Bleeding Air From The Hydraulic System 6-3

Jack Descent Rate Adjustment 6-4

Caster Assembly Replacement 6-4

Caster Cover Installation And Removal 6-5

Steer Caster Linkage Adjustment 6-6

Brake Adjustment 6-7

110V Power Cord Replacement 6-7

Lift Potentiometer Replacement 6-8

Lift Potentiometer "Burn-In" Procedure 6-9

Coil And Wire Replacement 6-10

Manual Release Valve Replacement 6-11

Solenoid Release Valve Replacement 6-11

Hydraulic Pump And Motor Replacement 6-12

Jack Replacement 6-13

Static Discharge Precautions

The electronic circuits in the 2035 are completely protected from static electricity damage only while the bed is assembled. It is extremely important that all service personnel always use adequate static protection when servicing the electronic systems of the 2035. *Whenever you are touching wires, you should be using static protection.*

Static Protection Equipment

The necessary equipment for proper static protection is:

- 1 static wrist strap; 3M part number 2214 or equivalent,
- 1 grounding plug; 3M part number 61038 or equivalent,
- 1 test lead with a banana plug on one end and an alligator clip on the other; Smith part number N132B699 or equivalent.

Stryker has available the following equipment for proper static protection:


- Complete static protection system – part number 3000–000–753
- 1 grounding plug – part number 3000–000–754
- 1 static wrist strap – part number 3000–000–755
- 1 test lead – part number 3000–000–756

CAUTION

All electronic service parts will be shipped in static shielding bags. Do not open the bags until you have completed steps 2 and 3 of the following procedure. Do not place unprotected circuit boards on the floor. All circuit boards to be returned to Stryker Medical should be shipped in the static shielding bags the new boards were shipped in.

Static Protection Procedure

1. Unplug the power cord from the wall receptacle.
2. Insert the grounding plug into a properly grounded hospital grade wall receptacle. Plug the banana plug of the test lead into the receptacle on the grounding plug. Connect the alligator clip on the other end of the test lead to a ground point on the bed.
3. Place the static control wrist strap on your wrist. Connect the alligator clip at the other end of the wrist strap cord to a ground point on the bed.

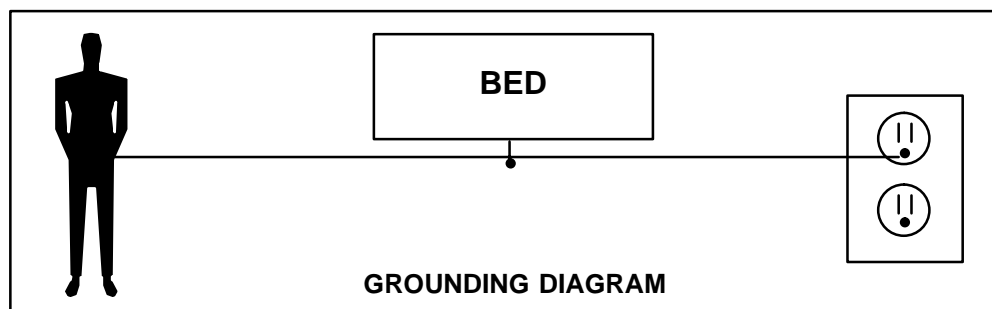

Checking Hydraulic Fluid Level

Required Tools:

3/16" Allen Wrench	3/4" Open End Wrench	(2) Sawhorses (or equivalent)
#2 Phillips Screwdriver	Bungee Cords (or equivalent)	Hydraulic Fluid

Procedure:

1. Raise the litter to the full up position.
2. Using a sawhorse or the equivalent, support the end of the litter needing service.
3. Using a 3/16" Allen wrench, remove the socket head cap screw from the litter support tube at the end of the bed needing service.
4. Press down on the release pedal for the end of the bed you are working on and, at the same time, push down on the jack actuator until it is fully down.
5. Using a #2 Phillips screwdriver, remove the three Phillips screws holding the base hood to the base frame.
6. If the bed is equipped with the 110V outlet option, remove the slotted screw holding the outlet cover. If the bed does not have the 110V outlet option, remove the two slotted screws holding the cover plate.
7. Using bungee cords or the equivalent, lift and support the base hood high enough to access the fill plug on the side of the jack.
8. Verify there are no hydraulic leaks. If there are, jack replacement will be necessary.
9. Using a 3/4" open end wrench, slowly turn the fill plug counterclockwise to allow excess system pressure to vent. Remove the fill plug.
10. The hydraulic fluid should be visible at the bottom of the fill hole. If it is not, add Mobil AW-32 hydraulic fluid (Stryker part number 1550-570-10) until the fluid is visible at the bottom of the fill hole. Replace the fill plug.



CAUTION

Use of other types of oil may damage hydraulic units.

11. Replace the base hood and reattach the litter support tube.

Bleeding Air From the Hydraulic System

NOTE

If the lift motion is jerky or sluggish when the "BED UP" button is pressed or when the lift pedal is pumped, and/or the litter does not hold when it is pushed down, there may be air in the hydraulic system. This may have been caused when the bed was unloaded from the shipping carton if it was lifted out by the litter instead of by the brake rings.

1. To bleed air from the hydraulic system, raise the litter to full up using the lift pedal and continue to pump 10-12 times after full up is reached.
2. Lower the bed using the foot release pedals. While continuing to hold the release pedals down, pump the lift pedal an additional 10-12 times.
3. Repeat the procedure 2-3 times or until the bed motion is smooth and the pedal action is firm.

Jack Descent Rate Adjustment

Required Tools:

Standard Screwdriver

#2 Phillips Screwdriver

Bungee Cords (or equivalent)

Adjustment Procedure:

1. Pump the litter up to full height.
2. Using a #2 Phillips screwdriver, remove the three Phillips screws holding the base hood to the base frame.
3. If the bed is equipped with the 110V outlet option, remove the slotted screw holding the outlet cover. If the bed does not have the 110V outlet option, remove the two slotted screws holding the cover plate.
4. Use bungee cords or the equivalent to secure the hood up and out of the way.
5. The descent rate needle valve is located on the top of the base of the jack. Turning the needle valve clockwise, with a screwdriver, will decrease the rate of descent. Turning it counterclockwise will increase the rate of descent.
6. Adjust the needle valve so the foot end of the stretcher descends slightly faster than the head end.

NOTE

The larger percentage of a patient's weight is located in the torso area. Adjust descent rate accordingly.

7. Remove the bungee cords supporting the base hood and secure the hood to the base frame.

NOTE

The jack descent rate was preset at the factory to drop the foot end faster than the head. It is recommended that the foot drop faster to avoid patient disorientation.

Caster Assembly Replacement

Required Tools:

#2 Phillips Screwdriver

Floor Jack

3/4" Wrench

Bungee Cords (or equivalent)

1" Wrench

Torque Wrench (w/ Ft. Lbs. Adjust.)


Standard Screwdriver

Replacement Procedure:

1. Pump the litter up to full height.
2. Using a #2 Phillips screwdriver, remove the three Phillips screws holding the base hood to the base frame.
3. If the bed is equipped with the 110V outlet option, remove the slotted screw holding the outlet cover. If the bed does not have the 110V outlet option, remove the two slotted screws holding the cover plate.
4. Use bungee cords or the equivalent to secure the hood up and out of the way.
5. Carefully remove the plastic wheel covers (see page 6–5).
6. Using a floor jack, lift the corner of the base frame approximately 4 inches off the floor.
7. While holding the cap screw with a 3/4" wrench, turn the lug nut with a 1" wrench to loosen the wheel assembly from the base frame. Remove the wheel.
8. Install the new wheel assembly and tighten the nut to 60 – 65 foot-pounds torque.
9. Lower the floor jack.
10. Install the plastic wheel covers onto the new wheel.
11. Remove the bungee cords supporting the base hood and secure the hood to the base frame.


Caster Cover Installation and Removal

1.

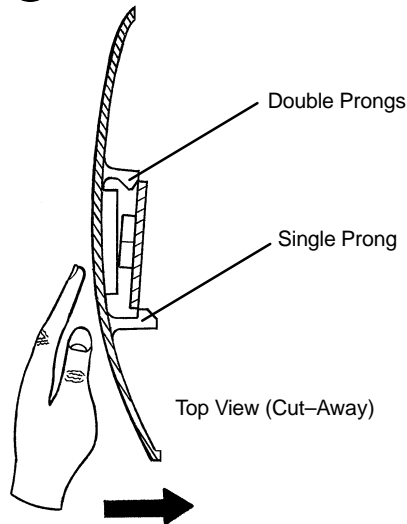


Looking through the larger of the two side cut-outs, align cover with axle nut or bolt head, as shown. Push down on the opposite side of the cover until single prong engages with caster horn.

2.

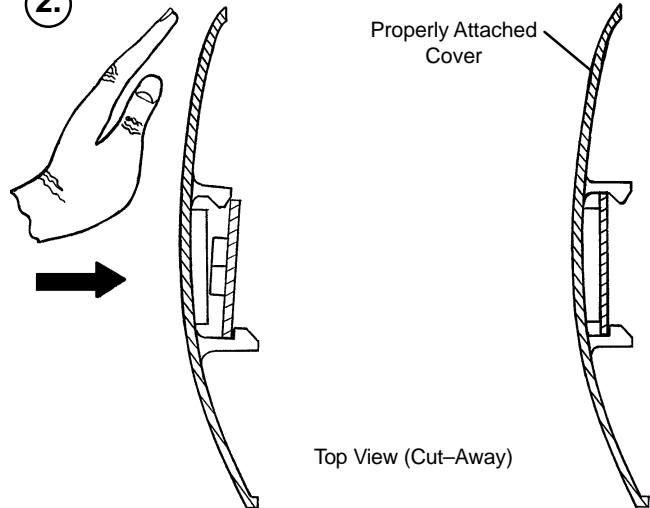

Push with palm on cover until double prongs engage.

3.

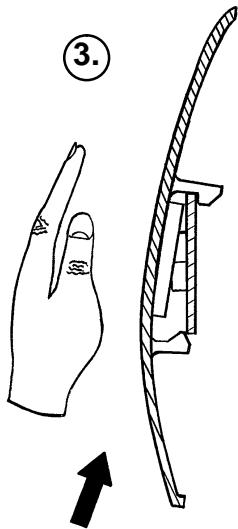

To remove wheel cover, insert large screwdriver into cut-out in side of wheel cover and into the space between the double prongs. Pry up cover to disengage double prongs and push sharply upward to disengage single prong.

Top View (Cut-Away)

Steer Caster Linkage and Adjustment

Required Tools:

#2 Phillips Head Screwdriver


3/8" Open End Wrench

5/32" Allen Wrench

1/2" Open End Wrench

Bungee Cords (or Equivalent)


Standard Screwdriver

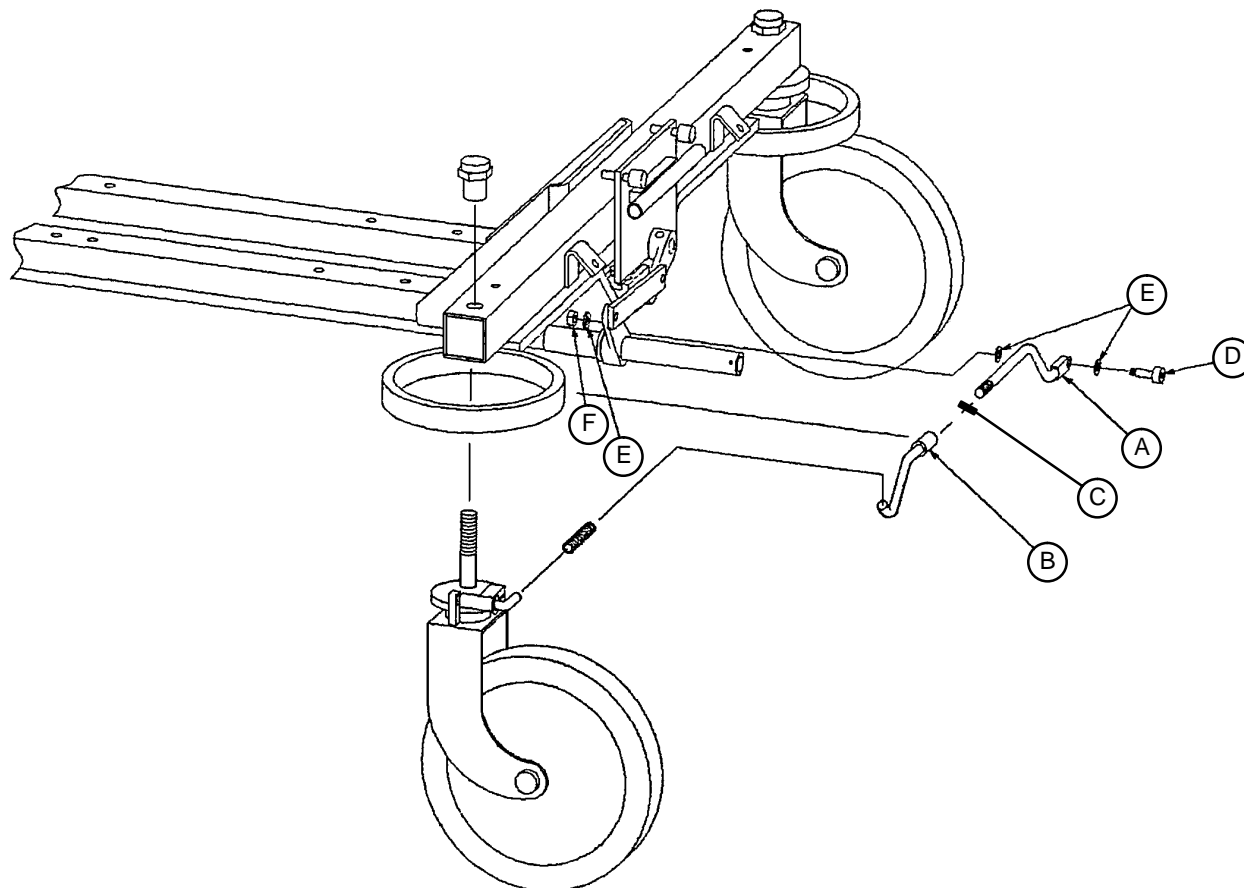

Adjustment Procedure:

1. Using a #2 Phillips screwdriver, remove the three Phillips screws holding the base hood to the base frame.
2. If the bed is equipped with the 110V outlet option, remove the slotted screw holding the outlet cover. If the bed does not have the 110V outlet option, remove the two slotted screws holding the cover plate.
3. Use bungee cords or the equivalent to secure the hood up and out of the way.
4. Using a 5/32" Allen wrench and 3/8" open end wrench, remove items (D), (E) and (F).
5. Using a 1/2" open end wrench, loosen jam nut (C).
6. Tighten item (A) into item (B) for more lever engagement or away from item (B) for less engagement.
7. To reassemble linkage, reverse steps 4 and 5.
8. Check steering wheel engagement. When properly adjusted, the locking tab should clear the notch in the steer caster when in the neutral position, and should fully engage when in the steer position.
9. After the linkage is properly adjusted, reinstall the base hood.

Brake Adjustment

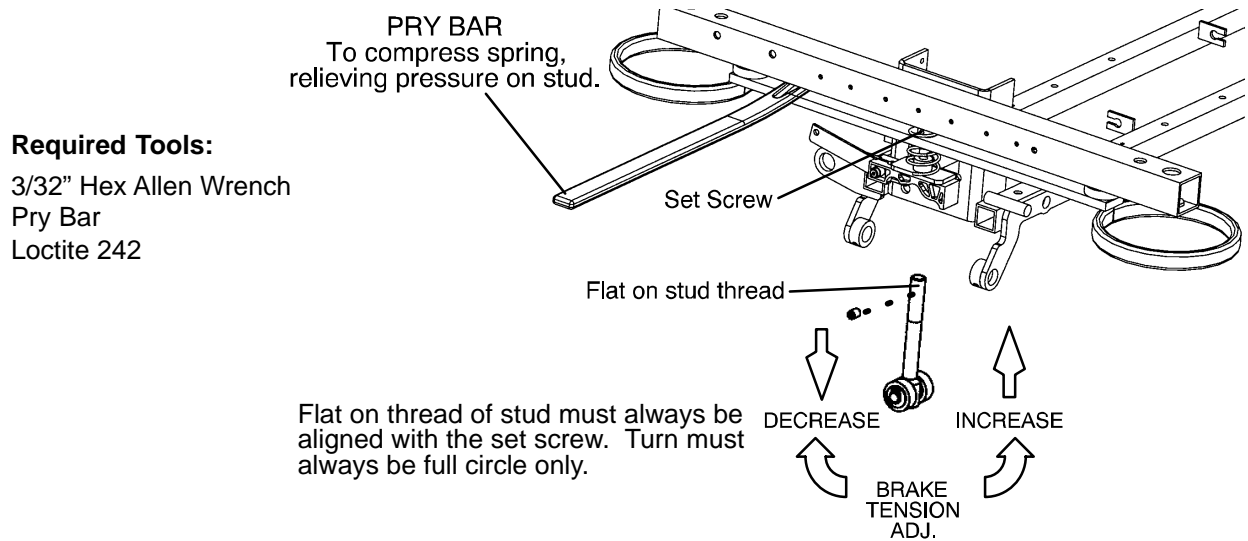

110V Outlet Power Cord Replacement

Required Tools:

#2 Phillips Screwdriver
 7/16" Socket Wrench

7/64" Allen Wrench
 Flat Blade Screwdriver

Bungee Cords (or Equivalent)

Replacement Procedure:

1. Raise the litter up to full height.
2. Unplug the power cord from the wall socket.
3. Using a #2 Phillips screwdriver, remove the three Phillips screws holding the base hood to the base frame.
4. Using a flat blade screwdriver, remove the slotted screw holding the outlet cover to the base hood and remove the cover.
5. Use bungee cords or the equivalent to secure the hood up and out of the way.
6. Remove the two 7/64" Allen screws holding the cover plate to the junction box and remove the cover.
7. Using a 7/16" socket wrench, remove the bolt from the ground ring terminal.
8. Unplug the power cord from the wiring harness and remove the power cord.
9. Reverse the procedure to install the replacement power cord and reinstall the base hood.

Lift Potentiometer Replacement

Required Tools:

#2 Phillips Screwdriver
1/2" Open End Wrench

Standard Screwdriver
Wire Cutters

Bungee Cords (or Equivalent)
Small Standard Screwdriver

Replacement Procedure:

1. Raise the litter up to full height and unplug the bed power cord from the wall socket.
2. Using a #2 Phillips screwdriver, remove the three Phillips screws holding the base hood to the base frame.
3. If the bed is equipped with the 110V outlet option, remove the slotted screw holding the outlet cover. If the bed does not have the 110V outlet option, remove the two slotted screws holding the cover plate.
4. Use bungee cords or the equivalent to secure the hood up and out of the way.
5. Using a 1/2" open end wrench, remove the nut holding the potentiometer to the bracket.
6. Remove the potentiometer by pulling it outward.

NOTE

The nylon gear will slide off the end of the shaft.

7. Using wire cutters, remove the cable ties holding the cable to the bracket and frame, unplug the cable, and remove the potentiometer.
8. Reverse steps 5–7 to install the replacement potentiometer.

NOTE

After a new lift potentiometer has been installed, a "burn-in" procedure must be done. See page 6–9 for the lift potentiometer "burn-in" procedure.

Lift Potentiometer "Burn-In" Procedure

1. After a new potentiometer has been installed, it must be calibrated. (If a new potentiometer was not installed, go to step 2.) Use a small standard screwdriver to reach through the access hole in the potentiometer bracket into the slot in the end of the potentiometer shaft. Turn the potentiometer shaft clockwise until it stops and then back counterclockwise 1 1/2 turns.
2. Press and hold the "Bed Motion Lock" and "Knee Lockout" buttons on the foot board.
3. While continuing to hold the two buttons down, plug the bed power cord into the wall socket. Release the two buttons and the Siderail Control Lights LED should be flashing. This indicates the bed is in the correct calibration mode.

NOTE

If your bed is equipped with a scale system, the display will read "LO HT SELECTED!" or "HI HT SELECTED!". This indicates whether the bed is a low height or a high height style bed. If it is a low height style bed, use the procedure described in step 4. If it is a high height style bed, use the procedure described in step 5.

4. To "burn in" the Bed Up/Down limits for a low height bed, lower the bed to the full down position. Press and hold the "Bed Motion Lock" and the "Knee Down" buttons at the same time. The "Bed Motion Lock" LED will light. Continue to hold the "Bed Motion Lock" and "Knee Down" buttons until the "Bed Motion Lock" LED flashes. The flashing LED indicates the limits have been set. Release the "Bed Motion Lock" button first, then the "Knee Down" button.
Important: If the buttons are not released as described above, the bed could be "burned in" incorrectly. Unplug the bed power cord to complete the "burn in" mode.
5. To "burn in" the Bed Up/Down limits for a high height bed, lower the bed to the full down position. Press and hold the "Bed Motion Lock" button. The "Bed Motion Lock" LED will light. Continue to hold the "Bed Motion Lock" button until the "Bed Motion Lock" LED flashes. The flashing LED indicates the limits have been set. Release the button and unplug the bed power cord to complete the "burn in" mode.
6. Reattach the base hood and run the bed up and down a few times to verify proper operation.
7. The distance between the floor and the top of the litter seat section (without mattress) on a high height bed should be approximately 22 1/2" with the litter fully down and approximately 35" with the litter fully up. On a low height bed, the distances should be approximately 20 1/2 " to 31".

NOTE

This procedure is the same for the head and the foot end of the bed.

Coil and Wire Replacement (Head End)

Required Tools:

#2 Phillips Screwdriver	Standard Screwdriver	Bungee Cords (or Equivalent)
3/8" Nut Driver	7/64" Allen Wrench	

Replacement Procedure:

1. Raise the litter up to full height and unplug the bed power cord from the wall socket.
2. Using a #2 Phillips screwdriver, remove the three Phillips screws holding the base hood to the base frame.
3. If the bed is equipped with the 110V outlet option, remove the slotted screw holding the outlet cover. If the bed does not have the 110V outlet option, remove the two slotted screws holding the cover plate.
4. Use bungee cords or the equivalent to secure the base hood up and out of the way.
5. Using a 3/8" nut driver, remove the nut holding the coil and wire to the solenoid valve and remove the coil and wire from the valve.
6. Using a 7/64" Allen wrench, remove the two screws holding the cover on the power junction box and remove the cover.
7. Unplug and remove the wires from the junction box.
8. Reverse the above procedure to install a new coil and wire assembly and reinstall the base hood.

Coil and Wire Replacement (Foot End)

Required Tools:

#2 Phillips Screwdriver	Standard Screwdriver	Bungee Cords (or Equivalent)
3/8" Nut Driver	5/32" Allen Wrench	Large Vise Grip Pliers
Wire Cutters		

Replacement Procedure:

1. Raise the litter up to full height and unplug the bed power cord from the wall socket.
2. Using a #2 Phillips screwdriver, remove the three Phillips screws holding the base hood to the base frame.
3. If the bed is equipped with the 110V outlet option, remove the slotted screw holding the outlet cover. If the bed does not have the 110V outlet option, remove the two slotted screws holding the cover plate.
4. Use bungee cords or the equivalent to secure the hood up and out of the way.
5. Depress the pump pedal and hold it in the down position. Using large vise grip pliers, clamp the pump bar from the top so the pliers rest against the pump motor plate.

NOTE

Be sure the pliers are clamped tight before continuing with the next step.

6. Using a 5/32" Allen wrench, loosen the two set screws on the pump pedal pivot bar.
7. Push the pivot bar completely out of the pump pedal assembly and lower the pedal assembly to the floor.
8. Using a 3/8" nut driver, remove the nut holding the coil and wire assembly to the solenoid valve and remove the coil and wire from the valve.
9. Using wire cutters, cut the wire ties holding the wires to the frame and unplug the wires.
10. Reverse the above procedure to install the new coil and wire assembly

CAUTION

Do not remove the vise grips until the pump pivot bar is securely back in place. Damage to the equipment or personal injury could occur.

Manual Release Valve Replacement

Required Tools:

#2 Phillips Screwdriver	Standard Screwdriver	Bungee Cords (or Equivalent)
Sawhorses (or Equivalent)	3/32" Allen Wrench	7/8" Open End Wrench

Replacement Procedure:

1. Raise the litter up to full height and unplug the bed power cord from the wall socket.
2. Support both ends of the litter top with sawhorses or the equivalent.
3. Using a #2 Phillips screwdriver, remove the three Phillips screws holding the base hood to the base frame.
4. If the bed is equipped with the 110V outlet option, remove the slotted screw holding the outlet cover. If the bed does not have the 110V outlet option, remove the two slotted screws holding the cover plate.
5. Use bungee cords or the equivalent to secure the hood up and out of the way.
6. Using a 3/32" Allen wrench, loosen the set screw holding the release paddle to the pedal rod. Slide the paddle out of the way.
7. Using a 7/8" open end wrench, remove the valve. Remove the stop sleeve from the old valve. It will be used with the new valve.
8. Reverse the above procedure to install the new valve and reinstall the base hood.

Solenoid Release Valve Replacement

Required Tools:

#2 Phillips Screwdriver	Small Standard Screwdriver	1 1/8" Deep Well Socket Wrench
Bungee Cords (or Equivalent)	Sawhorses (or Equivalent)	3/8" Nut Driver
5/32" Allen Wrench	Large Vise Grip Pliers	Wire Cutters

Replacement Procedure:

1. Remove the coil and wire from the solenoid (see page 6–10).
2. Support both ends of the litter top with sawhorses or the equivalent.
3. Using a 1 1/8" deep well socket wrench, remove the valve.
4. Reverse the above procedure to install the replacement valve.

Hydraulic Pump and Motor Replacement

Required Tools:

#2 Phillips Screwdriver	Standard Screwdriver	11/16" Open End Wrench
Bungee Cords (or Equivalent)	Saw Horses (or Equivalent)	3/8" Socket Wrench
9/16" Open End Wrench		

Replacement Procedure:

1. Raise the litter up to full height and unplug the bed power cord from the wall socket.
2. Support both ends of the litter top with sawhorses or the equivalent.
3. Using a #2 Phillips screwdriver, remove the three Phillips screws holding the base hood to the base frame.
4. If the bed is equipped with the 110V outlet option, remove the slotted screw holding the outlet cover. If the bed does not have the 110V outlet option, remove the two slotted screws holding the cover plate.
5. Use bungee cords or the equivalent to secure the hood up and out of the way.
6. Using a 3/8" socket wrench, remove the three bolts holding the pump motor cover to the base and remove the cover.
7. Unplug the cables for the motor and capacitor, noting the location of each so they will be reconnected properly.
8. Using a 11/16" open end wrench, remove the two return line fittings from the pump.
9. Using a 9/16" open end wrench, remove the two pressure line fittings from the pump. Note the location of each fitting so they will be reconnected properly.
10. Using a 3/8" socket wrench, remove the four bolts holding the pump motor to the brackets and remove the motor.
11. Reverse the above procedure to install the new pump and motor assembly and reinstall the base hood.
12. Run the bed up and down a few times to verify proper operation before returning it to service.

Jack Replacement

Required Tools:

#2 Phillips Screwdriver	Small Standard Screwdriver	Bungee Cords (or Equivalent)
Sawhorses (or Equivalent)	3/16" Allen Wrench	3/8" Socket Wrench
9/16" Open End Wrench	11/16" Open End Wrench	1/2" Open End Wrench
1/2" Socket Wrench	Floor Jack	1/4" Allen Wrench
Wire Cutters		

Replacement Procedure:

1. Raise the litter up to full height and unplug the bed power cord from the wall socket. If the bed will not raise electrically or manually, pull up on the litter top until it is fully raised.
2. Support both ends of the litter top with sawhorses or the equivalent.
3. Using a #2 Phillips screwdriver, remove the three Phillips screws holding the base hood to the base frame.
4. If the bed is equipped with the 110V outlet option, remove the slotted screw holding the outlet cover. If the bed does not have the 110V outlet option, remove the two slotted screws holding the cover plate.
5. Use bungee cords or the equivalent to secure the hood up and out of the way.
6. Using a 3/16" Allen wrench, remove the socket head cap screw from the litter support tube at the end of the bed needing service.
7. While depressing the appropriate release pedal, separate the jack shaft from the support tube and depress the jack shaft to the full down position.
8. Using a 1/4" Allen wrench and a 1/2" socket wrench, remove the nut and bolt holding the pump piston to the pump bar.

NOTE

If you are working on the foot end of the bed, be careful when removing the bolt. The spring must be compressed and the pump bar bracket must be pushed down and out of the way so the bolt and the spring can be removed.

9. Using wire cutters, remove all wire ties holding the pressure and return lines to the frame.
10. Using a 3/8" socket wrench, remove the three bolts holding the pump motor cover to the base and remove the cover.
11. Using a 9/16" wrench for the pressure line and a 11/16" wrench for the return line, remove the lines for the jack being replaced at the pump.
12. Using a 1/2" wrench and a 1/2" socket wrench, remove the two nuts and bolts holding the jack support bracket on the jack and remove the bracket.
13. Using a 1/2" wrench and a 1/2" socket wrench, remove the four nuts and bolts holding the jack assembly to the frame. Using a floor jack, lift the end of the bed you are servicing and remove the jack from underneath the base frame.
14. Reverse the above procedure to install the new jack and reinstall the base hood.

GENERAL INFORMATION

This section contains tool lists and step-by-step procedures to assist with the maintenance and servicing of the litter portion of your equipment.

In the text, the words "right" and "left" refer to the right and left sides of a patient lying face up on the bed.

LITTER MAINTENANCE CONTENTS

Scale System Diagnostics and Calibration 7-2 – 7-4

Load Cell Replacement 7-4

Head Motor Removal and Replacement 7-5

Knee Motor Removal and Replacement 7-6

Power Supply Removal and Replacement 7-7

CPU Board Removal and Replacement 7-7

Fowler and Lift Potentiometer "Burn-In" Procedure 7-8

Optional Smart TV Interface "Burn-In" Procedure 7-9

Scale System Diagnostics and Calibration

Diagnostic Mode Functions:

1. **Calibrate Scale:** This is required in the field if a CPU board or a load cell is replaced.
2. **Display Corner:** This function displays the individual reference numbers for each load cell assembly and can be used to isolate a defective load cell.
3. **Init. to Defaults:** This may be required in the field when replacing a CPU board or a load cell.
4. **Erase E-prom:** This is a factory setting not used in the field.
5. **Display Factor:** This is a factory setting not used in the field.
5. **Exit Diagnostics:** Changes made in the diagnostic mode **must** be saved in permanent memory using this function. Switching off power without saving will reset all variables to their previous values.

Diagnostic Mode:

NOTE

It requires **two people** to enable the diagnostic mode for the scale system.

1. To enter diagnostic mode, unplug the bed's power cord from the wall socket.
2. Press and **hold down** the LBS/KGS button.
3. While still holding the LBS/KGS button, plug the bed's power cord into the wall socket.
4. After two seconds, release the LBS/KGS button. The LCD should display "CALIBRATE SCALE". The diagnostic mode is now active.

Displaying Individual Load Cell Outputs:

A defective load cell can be detected by entering diagnostics and displaying individual load cell outputs.

1. Enter the diagnostic mode. The LCD will display "CALIBRATE SCALE" when the diagnostic mode is activated.
2. Repeatedly press and release the up or down arrow button (ZERO or SCALE ON/OFF) until the LCD displays "DISPLAY CORNER".
3. Press and release the ENTER button (LBS/KGS). The LCD should display "PICK CORNER NOW".

The four buttons listed in the group below function as POSITION buttons corresponding with the four corners of the bed's litter. Whenever the LCD displays "PICK CORNER NOW", press one of these buttons to select the load cell assembly at the desired corner.

- A. ZERO = head end, patient's left side
 - B. CHANGE EQUIP. = head end, patient's right side
 - C. TREND/FOWLER ANGLE = foot end, patient's right side
 - D. LBS/KGS = foot end, patient's left side
4. Press and release the position button that corresponds with the load cell to be checked. The LCD should display "X/X=NNN.N". "X/X" represents the initials of the selected corner, i.e. H/R will be displayed for the patient's head end, right side. "NNN.N" represents the resistance of the load cell.
 5. Repeat step four for each corner. Head end weight readings will normally be lower than foot end weights. Weight readings should be constant. A drifting 000.0 or 999.9 weight, or a reading that does not change when weight is applied to that corner of the bed indicates a problem with the selected load cell assembly or load cell cable.
 6. When all the load cell outputs have been checked, repeatedly press and release the SCALE ON/OFF button until the LCD displays "EXIT DIAGNOSTICS". Press the ENTER button (LBS/KGS) to exit diagnostics.

Scale System Diagnostics and Calibration

Verifying Scale Accuracy:

1. Zero the empty bed. Place a known weight on the center of the bed; the heavier the better and no less than 100 pounds. The displayed weight should be within 1% of the actual weight.
2. If the displayed weight is not accurate, remove the weight from the bed and proceed to the Scale Calibration section.

Scale Calibration:

NOTE

It requires **two people** to enable the calibration mode for the scale system.

Calibrate the scale system with a known 200 pound weight. If exactly 200 pounds is not available, the factory default for calibration will have to be changed as described in step 8.

1. To enter the calibration mode, unplug the bed's power cord from the wall socket.
2. Press and **hold down** the LBS/KGS button.
3. While still holding the LBS/KGS button, plug the bed's power cord into the wall socket.
4. After two seconds, release the LBS/KGS button. The LCD should read "CALIBRATE SCALE". The calibration mode is now active.
5. Using the up or down arrow button (ZERO or SCALE ON/OFF), toggle through the menu until "INIT. TO DEFAULTS" is displayed. Press and hold the ENTER button (LBS/KGS) until "SAVING DEFAULTS" is displayed. Release the ENTER button.
6. Using the up or down arrow buttons (ZERO or SCALE ON/OFF), toggle through the menu until "CALIBRATE SCALE" is displayed.
7. Press and hold the ENTER button (LBS/KGS). Zero the bed, following the displayed instructions. When the bed is zeroed, the LCD should display "REF X100=<2 0000". This is the factory default for 200 pounds. If 200 pounds will be used to calibrate the scale, proceed to step 9.
8. If exactly 200 pounds is not available, change the display to match the weight you are using. Pressing the TREND/FOWLER ANGLE button will move the cursor position to the right. Pressing the up arrow (ZERO) button will increase the numbers. Pressing the down arrow (SCALE ON/OFF) button will decrease the numbers. Scroll through the numbers until they match the weight you will use for calibration.
9. Press and release the ENTER button (LBS/KGS). The LCD will display "ADD H/L, HIT ON". Place the known weight on the litter over the head, left load cell.
10. Press the SCALE ON/OFF button. The LCD will display "RELEASE TO CAL." Release the button. The display should read "DO NOT TOUCH BED".
11. The LCD will display "ADD H/R, HIT ON". Place the known weight on the litter over the head, right load cell.
12. Press the SCALE ON/OFF button. The LCD will display "RELEASE TO CAL." Release the button. The display should read "DO NOT TOUCH BED".
13. The LCD will display "ADD F/R, HIT ON". Place the known weight on the litter over the foot, right load cell.
14. Press the SCALE ON/OFF button. The LCD will display "RELEASE TO CAL." Release the button. The display should read "DO NOT TOUCH BED".
15. The LCD will display "ADD F/L, HIT ON". Place the known weight on the litter over the foot, left load cell.
16. Press the SCALE ON/OFF button, the LCD will display "RELEASE TO CAL." Release the button. The display should read "DO NOT TOUCH BED".

Scale System Diagnostics and Calibration

Scale Calibration (Continued):

17. After all four corners have been calibrated, the LCD will display “CENTER WT ON BED”.
18. Remove the weight from the foot, left corner and position it on the center of the bed.
19. Press and release the SCALE ON/OFF button and the LCD will display “PRESS REV. TREND”. Press the button with the Reverse Trendelenburg symbol (feet down/head up) until the bed reaches full reverse Trend. Release the button and the LCD will display “DO NOT TOUCH BED”.
20. The LCD will display “PRESS TREND.” Press the button with the Trendelenburg symbol (feet up/head down) until the bed reaches full trend. Release the button and the LCD will display “DO NOT TOUCH BED”.
21. The LCD will display “CALIBRATE SCALE”. This indicates the calibration procedure is complete.
22. Exit scale calibration by pressing the up arrow button (ZERO) until the LCD displays “EXIT DIAGNOSTICS”. Press the ENTER (LBS/KGS) button to exit.
23. Level the bed at a full up or full down position. Remove the weight and zero the bed.
24. Verify scale accuracy and functionality before returning the bed to service.

Load Cell Replacement

Required Tools:

9/16” Socket Wrench
Wire Cutters

9/16” Open End Wrench

Saw Horse (or Equivalent)

Replacement Procedure:

1. Raise the Fowler or knee section, depending which end of the litter needs service.
2. Unplug the load cell connector from the load cell cable.
3. Using wire cutters, remove the wire ties holding the cable to the frame.
4. Using a 9/16” socket and a 9/16” open end wrench, remove the two bolts holding the load cell to the litter cross tube and remove the load cell.
5. Using a saw horse, support the litter at the end where the load cell was removed. Reverse the above procedure to install the new load cell.

NOTE

Scale calibration procedure must be performed after the load cell is replaced (see page 7–3).

Head Motor Removal and Replacement

Required Tools:

T27 Torx

7/16" Socket Wrench

3/8" Socket Wrench

Wire Cutters

Procedure:

1. Run the litter to the full up position and remove the mattress from the bed.
2. Fold the foot section back toward the head end of the bed. Electrically run the knee section to full up. If the knee section will not move electrically, pull the foot section toward the head end of the bed while pulling the CPR release handle (located at the head end of the bed). The knee section will raise.
3. Using a T27 torx, remove the four screws holding the cover to the actuator box and remove the cover.
4. Remove the two CPR release cables from the CPR release bracket. Using a 3/8" socket wrench underneath the actuator box, remove the two bolts holding the release bracket to the actuator box and remove the bracket from the actuator box.
5. Disconnect all the electrical connections going to the head motor and move aside any wiring that could interfere with the removal of the motor.
6. Using a 3/8" socket wrench underneath the actuator box, remove the four bolts holding the motor mounting bracket to the actuator box. Lift up and out on the motor to remove it.
7. Remove the motor mounting bracket from the old motor and install it on the replacement motor.
8. Reverse steps 3 through 6 to install the replacement motor.
9. Verify the bed is working properly before returning it to service.

Knee Motor Removal and Replacement

Required Tools:

T27 Torx

7/16" Socket Wrench

3/8" Socket Wrench

Wire Cutters

Procedure:

1. Run the litter to the full up position and remove the mattress from the bed.
2. Fold the foot section back toward the head end of the bed. Electrically run the knee section to full up. If the knee section will not move electrically, pull the foot section toward the head end of the bed while pulling the CPR release handle (located at the head end of the bed). The knee section will raise.
3. Using a 7/16" socket wrench, remove the mounting bolt on the litter for the knee dampening cylinder. This leaves the knee dampener mounted only to the seat panel.
4. Using a T27 torx, remove the four screws holding the cover to the actuator box and remove the cover.
5. Remove the two CPR release cables from the CPR release bracket. Using a 3/8" socket wrench underneath the actuator box, remove the two bolts holding the release bracket to the actuator box and remove the bracket from the actuator box.
6. Disconnect all the electrical connections going to the knee motor and move aside any wiring that could interfere with the removal of the motor.
7. Pull the foot panel toward the head end of the bed. This causes the knee motor linkage to roll past center and allows the motor to be removed without supporting the knee section.
8. Using a 3/8" socket wrench underneath the actuator box, remove the four bolts holding the motor mounting bracket to the actuator box. Lift up and out on the motor to remove it.
9. Remove the motor mounting bracket from the old motor and install it on the replacement motor.
10. Install the replacement motor.
11. Reverse step 3 – 5 to reinstall the knee dampener, CPR bracket and actuator box cover.
12. Pull the foot panel toward the foot end of the bed. This causes the knee motor linkage to roll back past center. If this step is not done, damage to the motor or linkage will occur.
13. Verify the bed is working properly before returning it to service.

Power Supply Removal and Replacement

Required Tools:

T27 Torx

Needle–Nose Pliers

Procedure:

1. Run the litter to the full up position and remove the mattress from the bed.
2. Fold the foot section back toward the head end of the bed. Electrically run the knee section to full up. If the knee section will not move electrically, pull the foot section toward the head end of the bed while pulling the CPR release handle (located at the head end of the bed). The knee section will raise.
3. Using a T27 torx, remove the four screws holding the cover to the actuator box and remove the cover.
4. Properly ground yourself (see page 6–2)
5. Unplug all electrical connections from the power supply.
6. Using needle–nose pliers, squeeze the four stand–offs supporting the power supply and pull up gently on the power supply to remove it.
7. Reverse steps 2 through 5 to install the new power supply.
8. Verify the bed is working properly before returning it to service.

CPU Board Removal and Replacement

Required Tools:

T27 Torx

Needle–Nose Pliers

Replacement Procedure:

1. Run the litter to the full up position and remove the mattress from the bed.
2. Fold the foot section back toward the head end of the bed. Electrically run the knee section to full up. If the knee section will not move electrically, pull the foot section toward the head end of the bed while pulling the CPR release handle (located at the head end of the bed). The knee section will raise.
3. Using a T27 torx, remove the four screws holding the cover to the actuator box and remove the cover.
4. Properly ground yourself (see page 6–2)
5. Unplug all electrical connections from the CPU board.
6. Press the six stand–offs away from the board while gently lifting the board up and out.
7. Install the replacement CPU board.

NOTE

After the replacement CPU board is installed, a “burn–in” procedure must be performed for the Fowler and lift motor potentiometers (see page 7–8)

If the bed is equipped with a scale system, a scale calibration procedure must also be performed after the replacement CPU board is installed (see page 7–3).

Fowler and Lift Potentiometer "Burn-In" Procedure

NOTE

It requires **two people** to enable the diagnostics mode for the bed.

1. Unplug the bed power cord from the wall socket.
2. Press and hold the "Bed Motion Lock" and "Knee Lockout" buttons on the foot board.
3. While continuing to hold the two buttons down, plug the bed power cord into the wall socket. Release the two buttons and the Siderail Control Lights LED should be flashing. This indicates the bed is in the correct calibration mode.
4. Using the foot board controls, run the Fowler up to 90°. Press and hold the button on the foot board to lock out the siderail controls for the back until the padlock LED flashes. Release the button.
5. Using the foot board controls, run the Fowler down to 0°. Press and hold the button on the foot board to lock out the siderail controls for the knee until the padlock LED flashes. Release the button..

NOTE

If your bed is equipped with a scale system, the display will read "LO HT SELECTED!" or "HI HT SELECTED!". This indicates whether the bed is a low height or a high height style bed. If it is a low height style bed, use the procedure described in step 6. If it is a high height style bed, use the procedure described in step 7.

6. To "burn in" the Bed Up/Down limits for a low height bed, lower the bed to the full down position. Press and hold the "Bed Motion Lock" and the "Knee Down" buttons at the same time. The "Bed Motion Lock" LED will light. Continue to hold the "Bed Motion Lock" and "Knee Down" buttons until the "Bed Motion Lock" LED flashes. The flashing LED indicates the limits have been set. Release the "Bed Motion Lock" button first, then the "Knee Down" button.

Important: If the buttons are not released as described above, the bed could be "burned in" incorrectly. Unplug the bed power cord to complete the "burn in" mode.

7. To "burn in" the Bed Up/Down limits for a high height bed, lower the bed to the full down position. Press and hold the "Bed Motion Lock" button. The "Bed Motion Lock" LED will light. Continue to hold the "Bed Motion Lock" button until the "Bed Motion Lock" LED flashes. The flashing LED indicates the limits have been set. Release the button and unplug the bed power cord to complete the "burn in" mode.
8. Plug the bed power cord into the wall socket and verify the back and bed lift limits are set properly before returning the bed to service.

NOTE

The distance between the floor and the top of the litter seat section (without mattress) on a high height bed should be approximately 22 1/2" with the litter fully down and approximately 35" with the litter fully up. On a low height bed, the distances should be approximately 20 1/2 " to 31".

Optional Smart TV Interface "Burn-In" Procedure

This procedure is used for selecting the style of TV interface desired for your bed. If traditional TV is desired, select the "No Flash" setting. If optional Smart TV is available on the bed, select one of the TV manufacturers listed in the table below.

SET-UP

- Ensure the communication cable is connected between the bed and the Db37 wall port or the pillow speaker port of the nurse call system. If available, a bed communication tester can be used instead of the hospital wiring.

PROCEDURE

1. Place the bed in the Fowler potentiometer burn-in mode (see page 7–8).
2. Notice the Nurse Call LED (yellow) is not flashing. This represents the traditional TV mode. Notice the Nurse Answer LED (green) is flashing on/off slowly.
3. Press and release the TV ON/OFF switch on the bed's siderail. Notice the Nurse Call LED flashes once. This is the first selection of TV manufacturers for the Smart TV mode. Notice the Nurse Answer LED (green) is flashing on/off slowly. The Nurse Answer LED will only light when the Nurse Call LED (yellow) is flashing.
4. Press and release the TV ON/OFF switch on the bed's siderail to scroll to other TV manufacturers. Notice the number of times the Nurse Call LED flashes matches the number listed in the table below and represents the TV manufacturer selected.
5. When the desired TV manufacturer has been selected, unplug the bed power cord from the wall socket and plug it back in to complete the Smart TV burn-in procedure.

NOTE

If the bed is connected to a television during the burn-in procedure, the television will turn on when the correct setting is selected.

TV MANUFACTURER SELECTION FOR SMART TV BURN-IN PROCEDURE		
Press and release TV ON/OFF switch:	Nurse Call LED (Yellow)	TV Manufacturer
No press	No flash	Traditional TV
Once	One flash	RCA 1
Twice	Two flashes	RCA 2
Three times	Three flashes	Zenith 1
Four times	Four flashes	Zenith 2
Five times	Five flashes	Phillips/Magnavox
Six Times	Six flashes	Magnavox (models 9120, 9220, 9320)

GENERAL INFORMATION

This section contains tool lists and step-by-step procedures to assist with the maintenance and servicing of the siderail portion of your equipment.

In the text, the words “right” and “left” refer to the right and left sides of a patient lying face up on the bed.

SIDERAIL MAINTENANCE CONTENTS

Head and Foot End Siderail Cover Removal 8-2


Head and Foot Molded Siderail Replacement 8-3

Head End Siderail Cable Replacement 8-4, 8-5

Head and Foot Siderail Cover Removal

Required Tools:

#2 Phillips Screwdriver

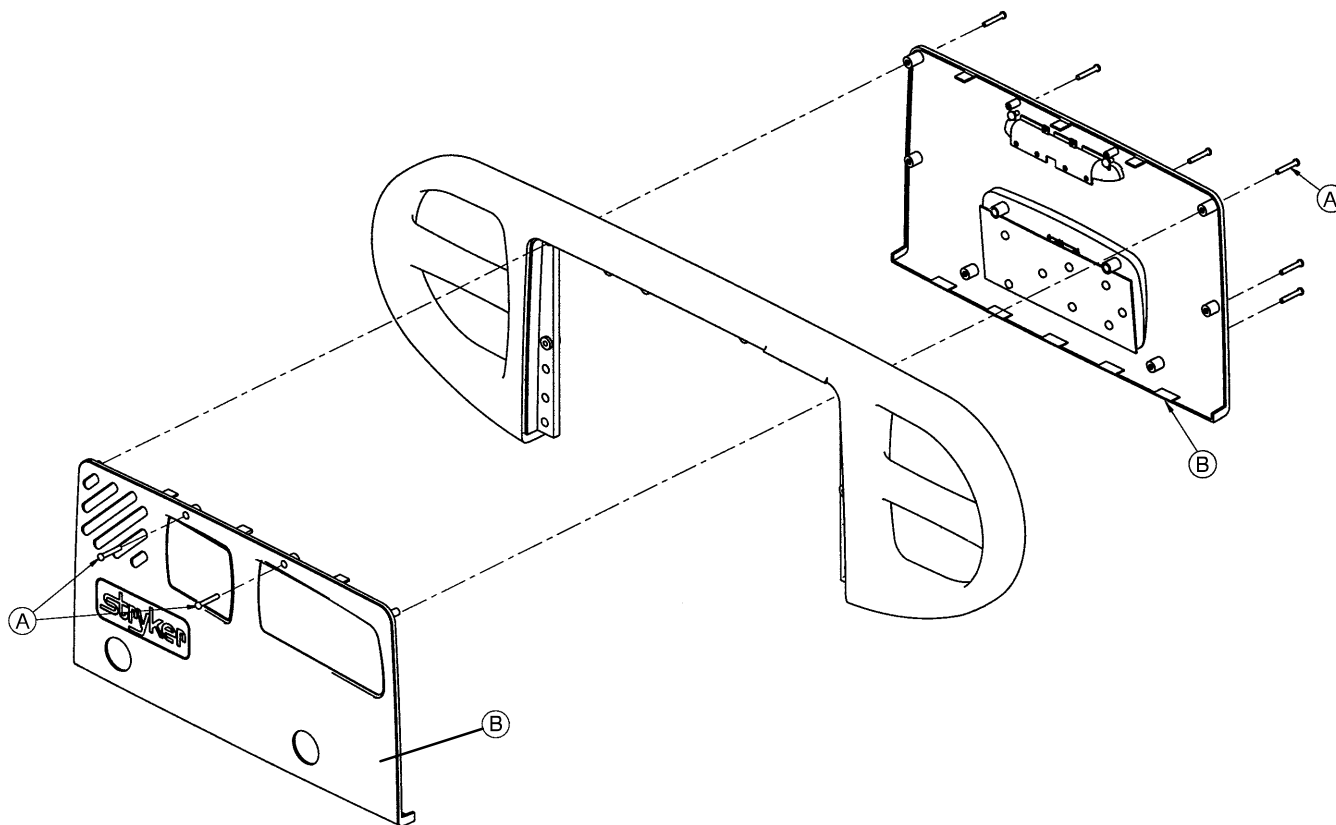

Removal Procedure:

1. Unplug the power cord from the wall receptacle.
2. Using a #2 Phillips screwdriver, remove the 8 phillips screws (A) holding the covers (B) to the siderail.

CAUTION

There are two cables connecting the outside cover to the head end siderail. Be careful not to pull on them when removing the cover.

3. Remove the cables from the siderail. Make note of the proper location for the cables.
4. Reverse the above steps to reattach the cover.

CAUTION

Do not snag the cables when installing the siderail cover.

NOTE


Follow the same procedure for siderail cover removal for the foot end rails.

Head and Foot Molded Siderail Replacement

Required Tools:

#2 Phillips Screwdriver

3/8" Nut Driver

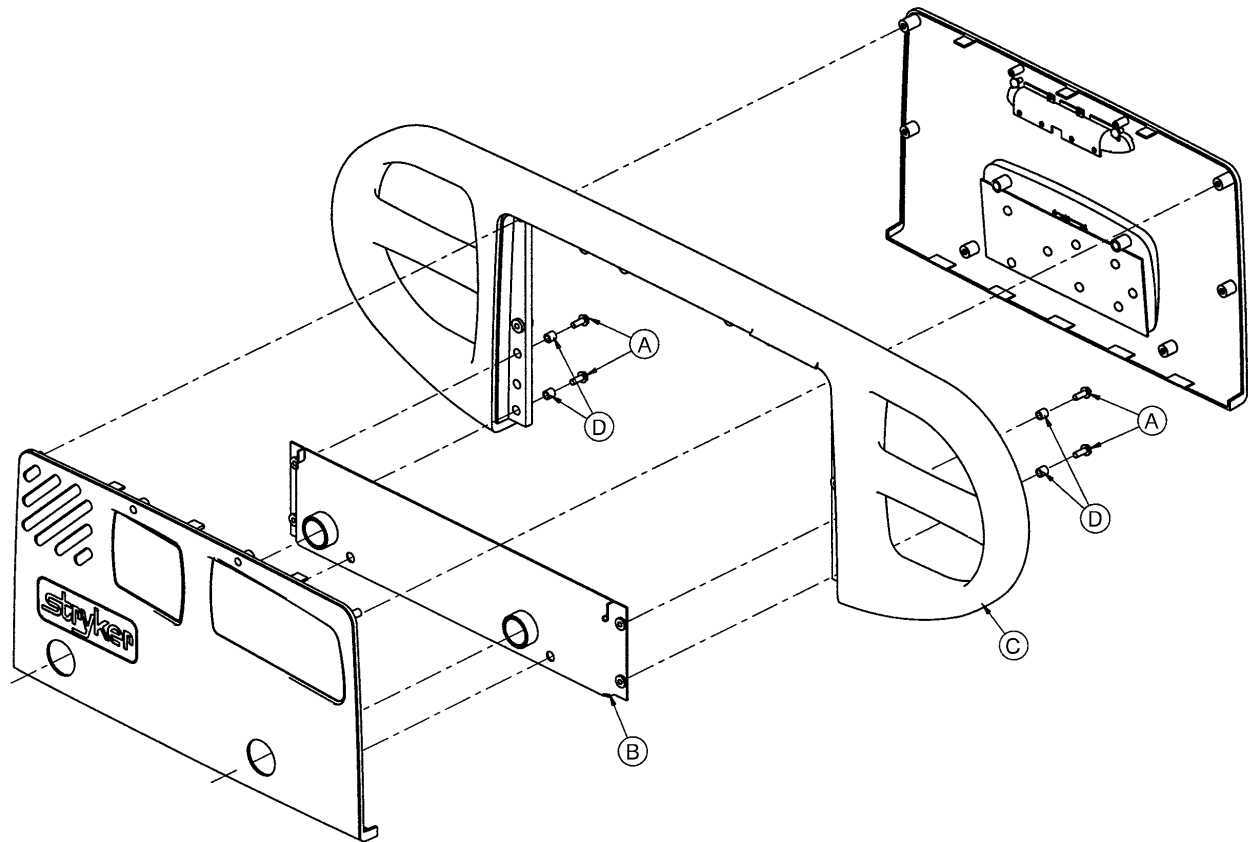**Procedure:**

1. Unplug the bed power cord from the wall socket.
2. Remove the siderail cover (see page 8-2).
3. Using a 3/8" nut driver, remove the four screws (A) holding the molded rail (C) to the siderail support assembly (B).

NOTE

Note the location of the spacers (D) for re-assembly purposes.


4. Pull up on the molded rail (C) to remove it from the siderail assembly.
5. Reverse the above steps to install the new molded rail.

Head End Siderail Cable Replacement

Required Tools:

#2 Phillips Screwdriver

Side Cutters

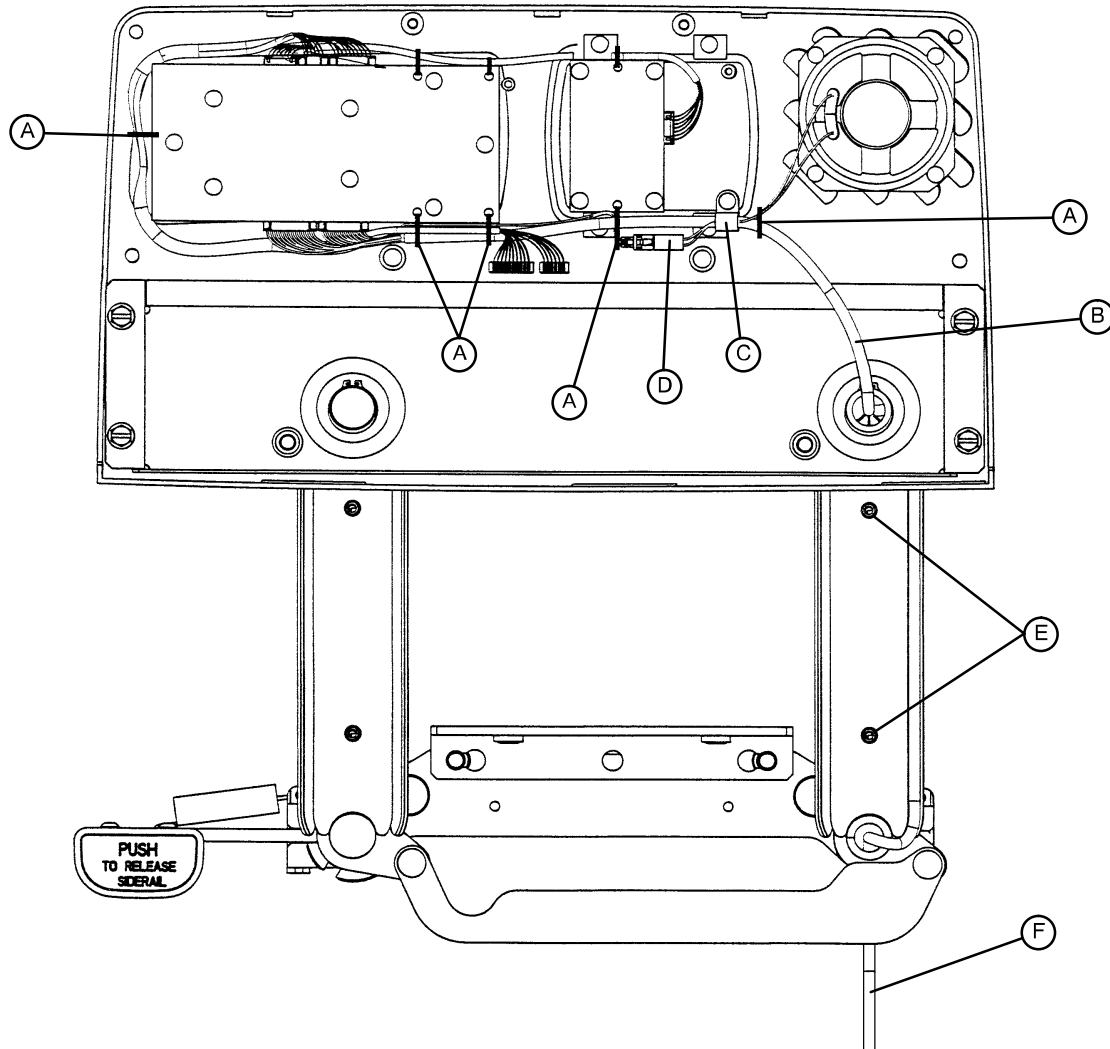

Procedure:

1. Run the head section fully up.
2. Unplug the bed power cord from the wall socket.
3. Using a #2 Phillips screwdriver, remove the eight screws holding the siderail cover and remove the cover.
4. Put the siderail in the down position.
5. Using a #2 Phillips screwdriver, remove the two screws (E) holding the rear siderail pivot arm cover to the pivot arm. Remove the cover to expose the siderail cables.

Head End Siderail Cable Replacement (Continued)

6. Using side cutters, clip the cable ties (A) holding the cables together.
7. Using a #2 Phillips screwdriver, remove the cable clamp (C) from the siderail.
8. Disconnect cable (B) from the circuit board and cable (D) from the speaker.

NOTE

The speaker and nurse call are optional equipment and may not be in the siderail as shown.

9. Pull the cables through the siderail (toward the center of the bed).
10. Unplug the cable assembly (F) underneath the head section.
11. Reverse the above steps to install the new cable.



CAUTION

Be sure to position the cables on both sides of the pivot arm, as shown in the illustration on page 8-4, before reattaching the pivot arm cover. If not done properly, the cover will not fit tightly and damage could occur to the cables.

GENERAL INFORMATION

This section contains tool lists and step-by-step procedures to assist with the maintenance and servicing of the foot board portion of your equipment.


FOOT BOARD MAINTENANCE CONTENTS

Foot Board Hinge Removal 9-2

Foot Board Module Replacement 9-3

Foot Board Interface Plug Replacement 9-4

Foot Board Hinge Removal

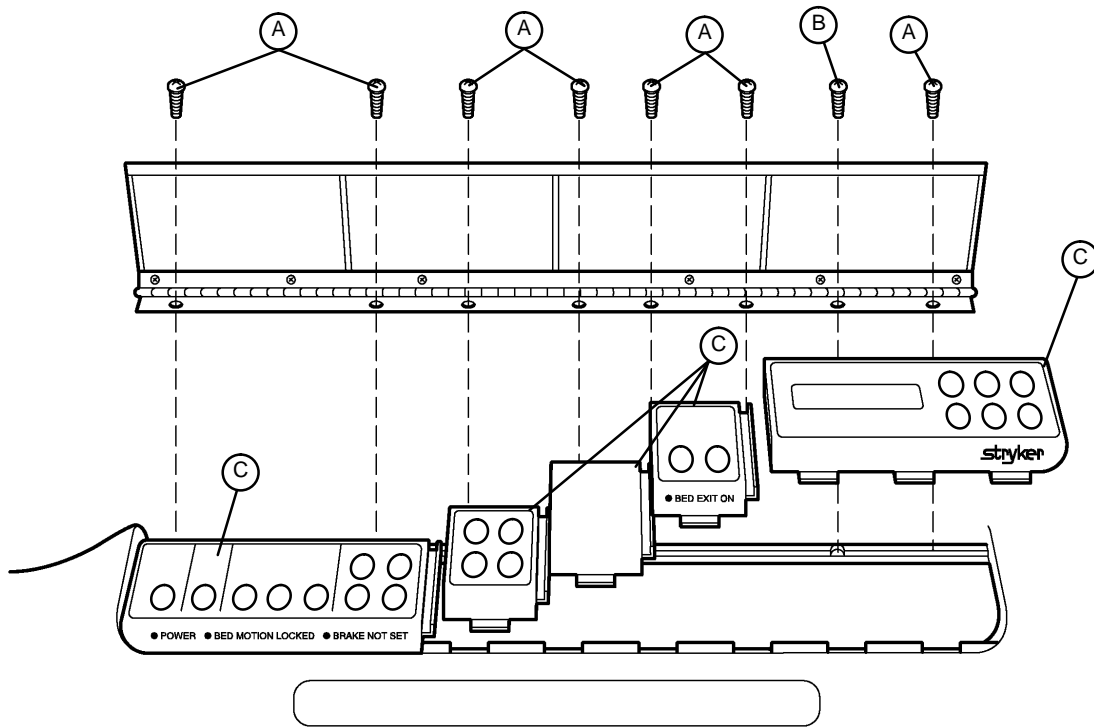

Required Tools:

#2 Phillips Screwdriver


Procedure:

1. Using a #2 Phillips screwdriver, remove the screws (A & B) holding the door and hinge assembly to the foot board.
2. If replacing the hinge only, use a Phillips screwdriver to remove the screws holding the hinge to the door.
3. Reverse the above steps to attach the replacement door and/or hinge.

NOTE

Screw (B) is a machine screw and must be reinstalled in the proper hole.

Foot Board Module Replacement

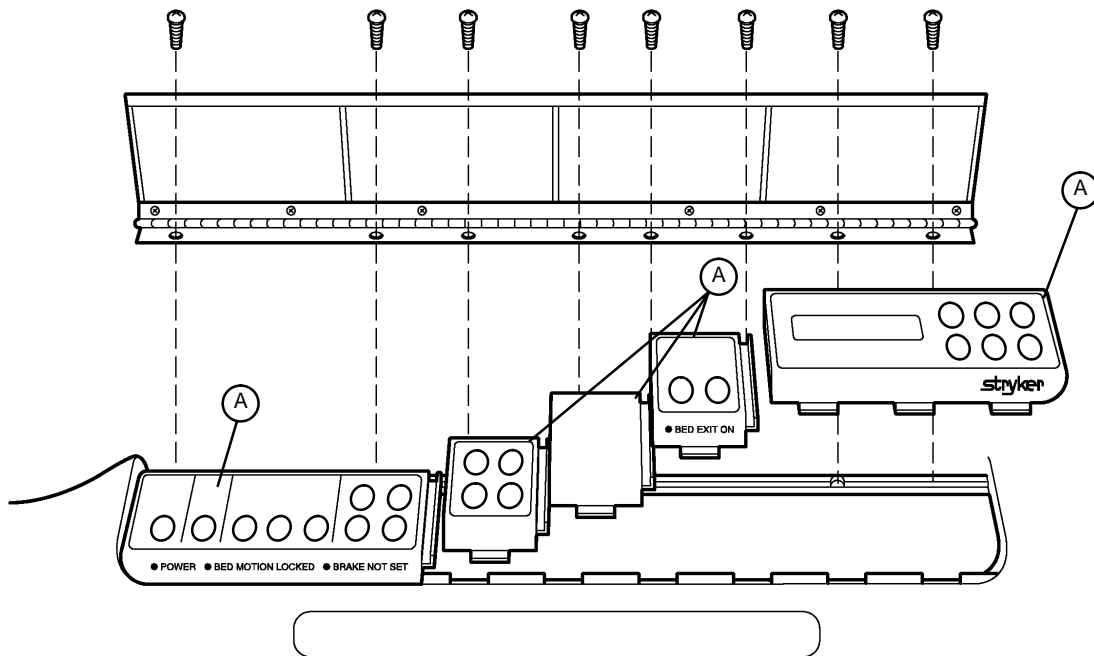

Required Tools:

#2 Phillips Screwdriver

Procedure:

1. Unplug the bed power cord from the wall socket. Remove the foot board hinge (see above).

NOTE

Regardless of which module is being replaced, the farthest module to the right must be removed first.

2. Pull the module out of the foot board and disconnect the cable from the module (A).
3. Reverse the above steps to install the new module.


 **CAUTION**

The modules must be overlapped as shown in the illustration to prevent fluids from entering the board cavity and causing damage.

Foot Board Interface Plug Replacement

Required Tools:

#2 Phillips Screwdriver

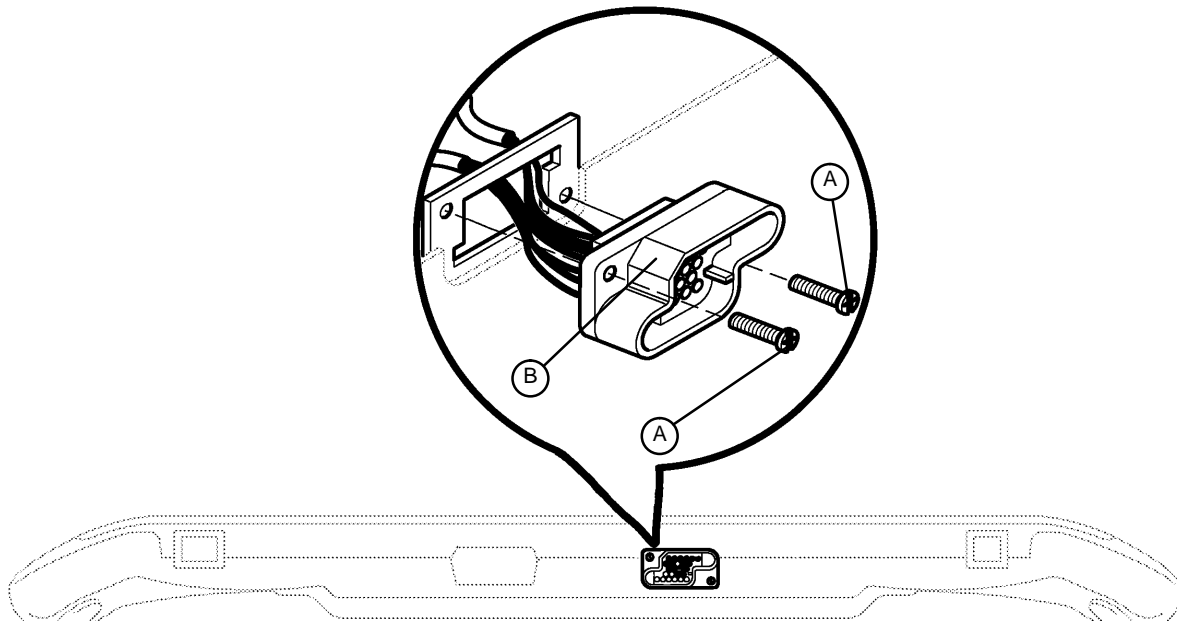

BOTTOM VIEW OF FOOT BOARD

Procedure:

1. Unplug the bed power cord from the wall socket.
2. Remove the foot board from the bed to access the bottom of the board.
3. Properly ground yourself (see page 6–2 for static discharge precautions).
4. Using a #2 Phillips screwdriver, remove the eight screws holding the foot board door to the foot board and remove the door.
5. Using a #2 Phillips screwdriver, remove the two screws (A) holding the plug to the foot board.
6. Disconnect the cable from the foot board module cable. Note proper placement of the cable so it will be reconnected properly.
7. Reverse the above steps to install the new interface plug.

CAUTION

Be sure to install the plug with the flat edge (B) at the top left, as shown in the illustration, or the foot board interface plug will not mate properly with the bed and damage to the plug or foot board could result.

GENERAL INFORMATION

This section contains tool lists and step-by-step procedures to assist with the maintenance and servicing of the optional Dynamic Mattress System.

In the text, the words “right” and “left” refer to the right and left sides of a patient lying face up on the bed.

DYNAMIC MATTRESS SYSTEM MAINTENANCE CONTENTS

Mattress Cover Replacement	10-2
Air Bladder Replacement	10-3
Foam Base Assembly Replacement	10-3
Control Unit Circuit Board Replacement	10-4
Control Unit Air Compressor Replacement	10-4, 10-5
Control Unit Replacement	10-5

Mattress Cover Replacement

Required Tools:

Wire Cutters

Static Protection Equipment (see page 6–2)

Replacement Procedure:

1. Unplug the power supply from the wall socket.
2. Unzip and remove the upper mattress cover.
3. Remove and set aside the upper foam layer.
4. Properly ground yourself. (See page 6–2 for static discharge precautions).
5. Unplug the cables from the control unit.
6. Remove the remaining foam assembly from the bottom cover and set aside.
7. Cut the cable ties securing the cables through the bottom cover grommet.
8. Remove the cables from the bottom cover.
9. Spread out the new bottom cover on a flat surface.
10. Replace the cables through the grommet in the bottom cover.
11. Place the foam assembly containing the air bladder and control unit into the bottom cover being sure the control unit is at the foot end of the new bottom cover.
12. Route the cables along the patient left side of the foam assembly to the control unit.
13. Reconnect the cables to the control unit.
14. Push the cables down into the slit in the base foam assembly.
15. Lift the head end of the foam assembly and pull the cable slack out of the mattress.
16. Fold the grommet material around the cable and replace the cable ties where they were located prior to disassembly.
17. Place the foam assembly back into the bottom cover.
18. Reinstall the upper foam layer with the *pink side down*.
19. Place the upper cover on the mattress with the “Head End” stamp at the head end of the mattress.
20. Zip the upper mattress cover onto the lower cover.
21. Plug the power supply into the wall socket and test the mattress for proper operation.

To replace the upper cover only:

1. Unzip and remove the old upper cover.
2. Unpack the new upper cover and lay it flat on the mattress, aligning the zipper properly.
3. Zip the upper cover onto the lower cover.

Air Bladder Replacement

Required Tools: None

Replacement Procedure:

1. Unplug the power supply from the wall socket.
2. Unzip and remove the upper mattress cover.
3. Remove and set aside the upper foam layer.
4. Unplug the hoses from the air bladder fittings.
5. Remove the air bladder from the mattress.
6. Install the new air bladder into the foam frame.
7. Install the hoses on the fittings of new air bladder.
8. Reinstall the upper foam layer, with the *pink side down*.
9. Reinstall the upper cover on the mattress with the “Head End” stamp at the head end of the mattress.
10. Zip the upper cover onto the lower cover.
11. Plug the power supply into the wall socket and test the mattress for proper operation.

Foam Base Assembly Replacement

Required Tools: Static Protection Equipment (see page 6–2)

Replacement Procedure:

1. Unplug the power supply from the wall socket.
2. Unzip and remove the upper mattress cover.
3. Remove and set aside the upper foam layer.
4. Disconnect the hoses from the air bladder. Remove the air bladder from the mattress and set it aside.
5. Properly ground yourself (see page 6–2 for static discharge precautions).
6. Disconnect the cables from the control unit. Remove the control unit and set it aside.
7. Remove entire foam base assembly.
8. Install the new foam base assembly with the cut–out for the control unit at the foot end of the cover.
9. Install the control unit into the new foam base.
10. Install the air bladder into the new base foam.
11. Reconnect the hoses to the fittings on the air bladder.
12. Route the cables along the patient left side of the foam assembly to the control unit.
13. Reconnect the cables to the control unit.
14. Push the cables down into the slit in the base foam assembly.
15. Reinstall the upper foam layer, *pink side down*.
16. Reinstall the upper cover on the mattress with the “Head End” stamp at the head end of the mattress.
17. Zip the upper cover onto the lower cover.
18. Plug the power supply into the wall socket and test the mattress for proper operation.

Control Unit Circuit Board Replacement

Required Tools:

Static Protection Equipment (see page 6–2) Standard Screwdriver Needle Nose Pliers
5/16" Socket Wrench or Nut Driver 3/16" Socket Wrench or Nut Driver

Replacement Procedure:

1. Unplug the power supply from the wall socket.
2. Unzip and remove the upper mattress cover.
3. Remove and set aside the upper foam layer.
4. Properly ground yourself (see page 6–2 for static discharge precautions).
5. Disconnect the cables from the control unit.
6. Disconnect the hoses from the air bladder.
7. Remove the control unit from the mattress.
8. Using a standard screwdriver, unsnap the cover of the control unit.
9. Unplug the power connector for the air compressor.
10. Remove the two 5/16" nuts holding the circuit board cover to the bottom cover of the control unit.
11. Remove the two 3/16" modified standoff screws holding the circuit board hand pendant port to the side of the bottom cover.
12. Remove the pressure switch hose from the circuit board.
13. Using needle nose pliers, squeeze the two standoffs holding the circuit board in place and lift up slightly on the circuit board from the circuit board end to remove the circuit board.
14. Reverse the above procedure to install the new circuit board.
15. Plug the power supply into the wall socket and test the mattress for proper operation.

Control Unit Air Compressor Replacement

Required Tools:

Standard Screwdriver

Replacement Procedure:

1. Unplug the power supply from the wall socket.
2. Unzip and remove the upper mattress cover.
3. Remove and set aside the upper foam layer.
4. Disconnect the cables from the control unit.
5. Disconnect the hoses from the air bladder.
6. Remove the control unit from the mattress.
7. Using a standard screwdriver, unsnap the cover of the control unit.
8. Unplug the power connector for the air compressor.
9. Unplug the pressure switch hose from the air compressor.

Control Unit Air Compressor Replacement (Continued)

10. Firmly grip the air compressor and lift it out of the control unit box (the air compressor is held down with double-faced tape).
11. Clean the double-faced tape off the bottom cover.
12. Remove the protective coating from the double-faced tape on the replacement air compressor.
13. Install the replacement air compressor into the control unit box. Press down firmly to secure the double-faced tape.
14. Reconnect the pressure switch hose to the air compressor.
15. Reconnect the power connector to the inside of the control unit box.
16. Snap the top cover onto the control unit.
17. Reinstall the control unit into the mattress.
18. Reconnect the hoses to the air bladder.
19. Reconnect the cables to the control unit.
20. Reinstall the upper foam layer.
21. Zip the upper cover onto the lower cover.
22. Plug the power supply into the wall socket and test the mattress for proper operation.

Control Unit Replacement

Required Tools:

Static Protection Equipment (see page 6–2)

Replacement Procedure:

1. Unplug the power supply from the wall socket.
2. Unzip and remove the upper mattress cover.
3. Remove and set aside the upper foam layer.
4. Properly ground yourself (see page 6–2 for static discharge precautions).
5. Disconnect the cables from the control unit.
6. Disconnect the hoses from the air bladder.
7. Remove the control unit from the mattress.
8. Install the new control unit into the mattress.
9. Connect the cables to the new control unit.
10. Reconnect the hoses to the air bladder.
11. Push the cables down into the slit in the base foam assembly.
12. Reinstall the upper foam layer, *pink side down*.
13. Reinstall the upper cover on the mattress with the “Head End” stamp at the head end of the mattress.
14. Zip the upper cover onto the lower cover.
15. Plug in the power supply and test the mattress for proper operation.

GENERAL INFORMATION

This section contains assembly drawings and parts lists to assist with the identification of individual components of the equipment and accessories.

In the parts lists, the words “right” and “left” refer to the right and left sides of a patient lying face up on the bed.

ASSEMBLY DRAWINGS AND PARTS LISTS CONTENTS

Base Assembly 11-3 – 11-14

Brake Adjuster Assembly 11-15

Brake Cam Assembly 11-16

Caster Assembly 11-17

Steer Caster Assembly 11-18

Wheel Assembly 11-19

Fifth Wheel Arm Assembly 11-20

Jack/Motor Assembly 11-22, 11-23

Jack Assembly 11-24, 11-25

Jack Base Assembly 11-26

Jack Pump Piston Assembly 11-27

Rack and Pinion Assembly 11-28, 11-29

Litter Assembly 11-30 – 11-42

Head End Siderail Assembly 11-43– 11-57

Head End Siderail Timing Link Assembly 11-58, 11-59

Head End Siderail Inner Panel Assembly 11-60

Head End Siderail Outer Panel Assembly 11-61

Head End Siderail DMS Module Assembly 11-62

Head End Siderail Smart TV Module Assembly 11-63, 11-64

Head End Siderail TV Control Module Assembly 11-65, 11-66

Foot End Siderail Assembly 11-67 – 11-70

Foot End Siderail Timing Link Assembly 11-71, 11-72

Head Board Assembly 11-73


Foot Board Assembly 11-74 – 11-79

Foot Board Main Module Assembly 11-80

ASSEMBLY DRAWINGS AND PARTS LISTS CONTENTS (CONTINUED)


Foot Board CPR Drop/Cardiac Chair Module Assembly	11-81
Foot Board Bed Exit Module Assembly	11-82
Pendant Assembly	11-83
Removable I.V. Pole Assembly	11-84
2-Stage I.V. Mounting Assembly, Foot End	11-85
2-Stage I.V. Mounting Assembly, Head End	11-86
2-Stage I.V. Mounting Assembly, Dual Head End	11-87
2-Stage I.V. Pole Assembly	11-88
I.V. Pole Latch Assembly	11-89
Fowler X-Ray Cassette Holder Assembly	11-90
Siderail Transducer Mount Assembly	11-91
I.V. Pole Transducer Mount Assembly	11-92
Upright Oxygen Bottle Holder Assembly	11-93
Defibrillator Tray Assembly	11-94
Optional Bed Extender Pad	11-95

Base Assembly

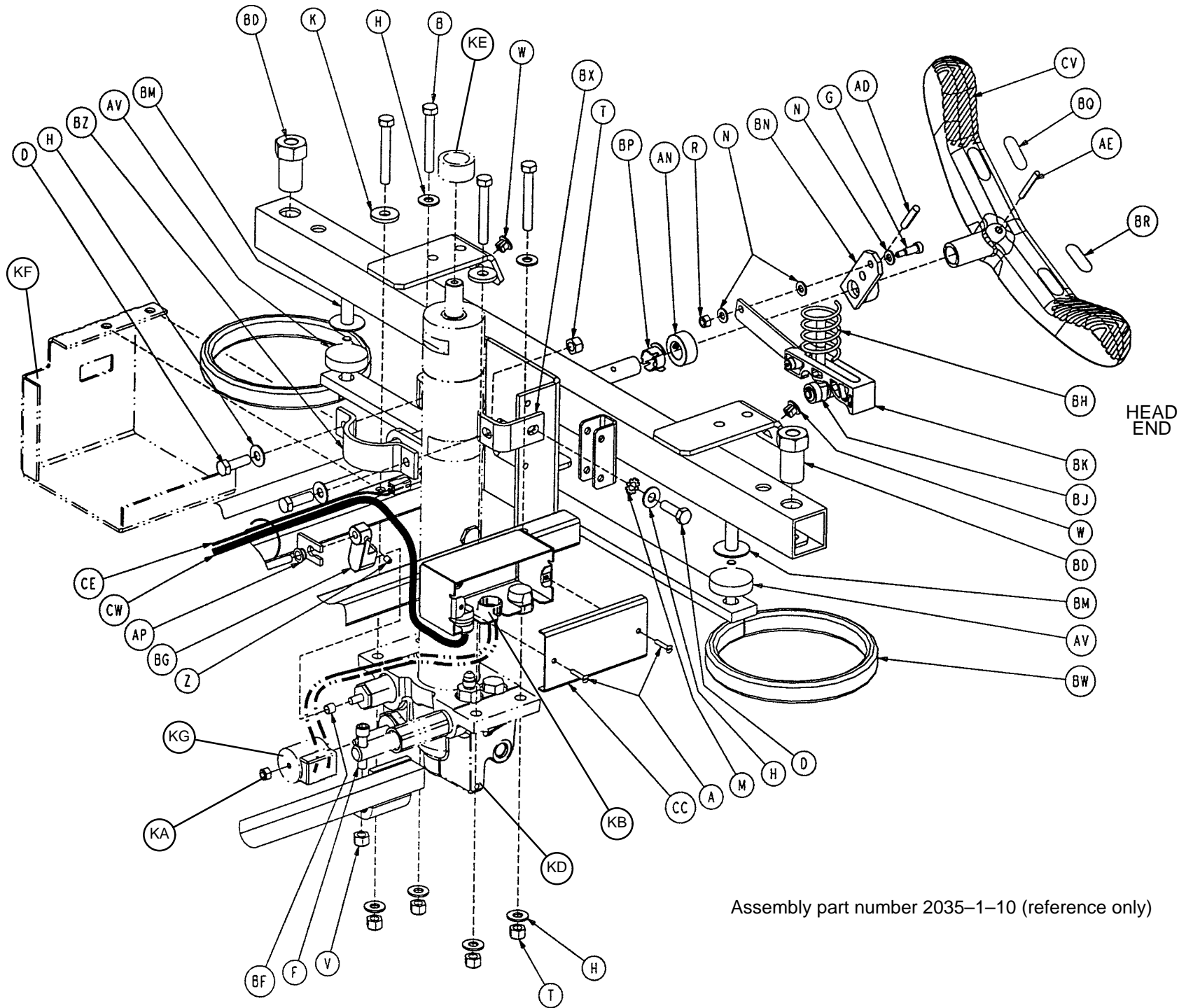


Assembly part number 2035-1-10 (reference only)

Base Assembly

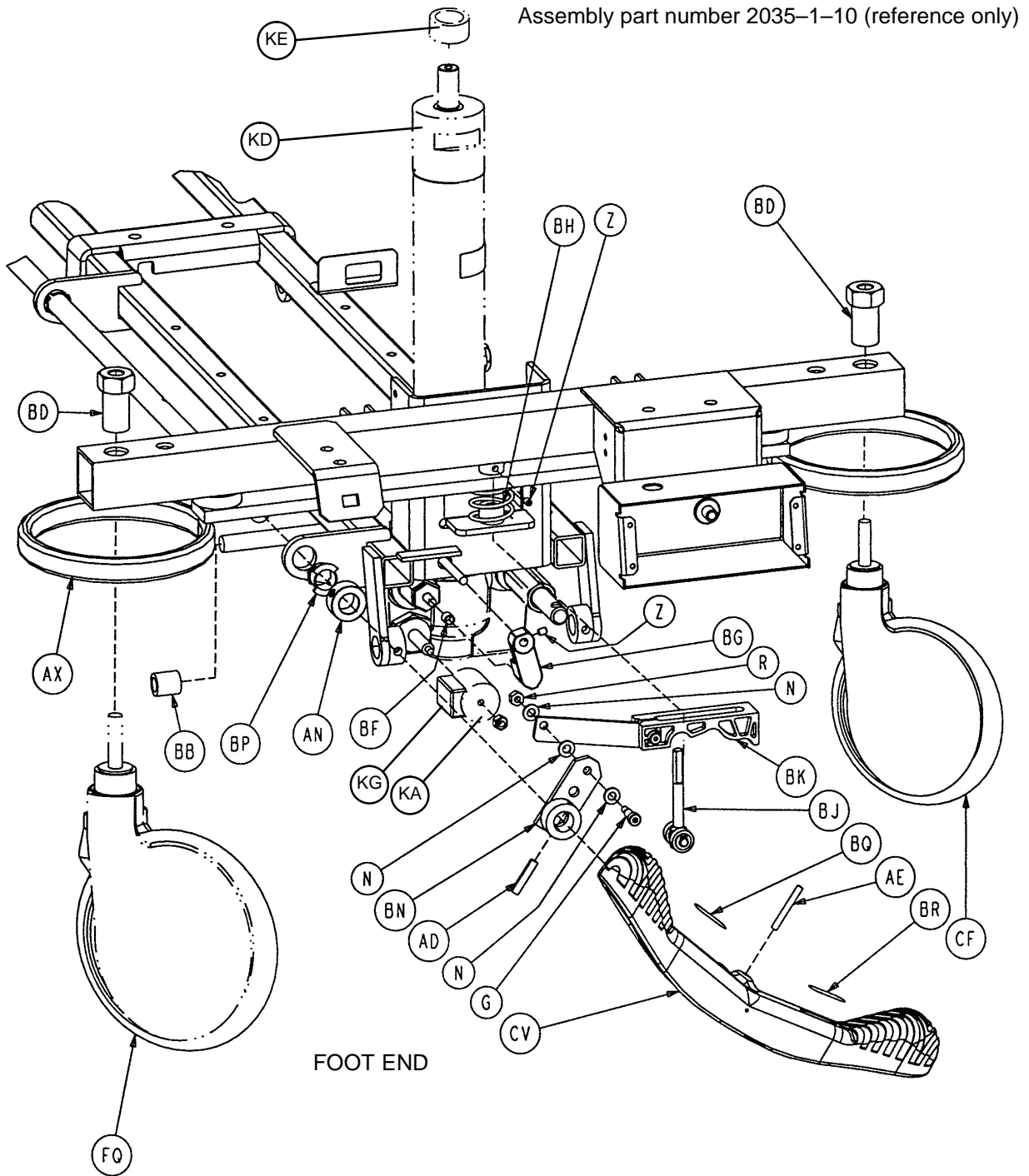


Base Assembly

Assembly part number 2035-1-10 (reference only)

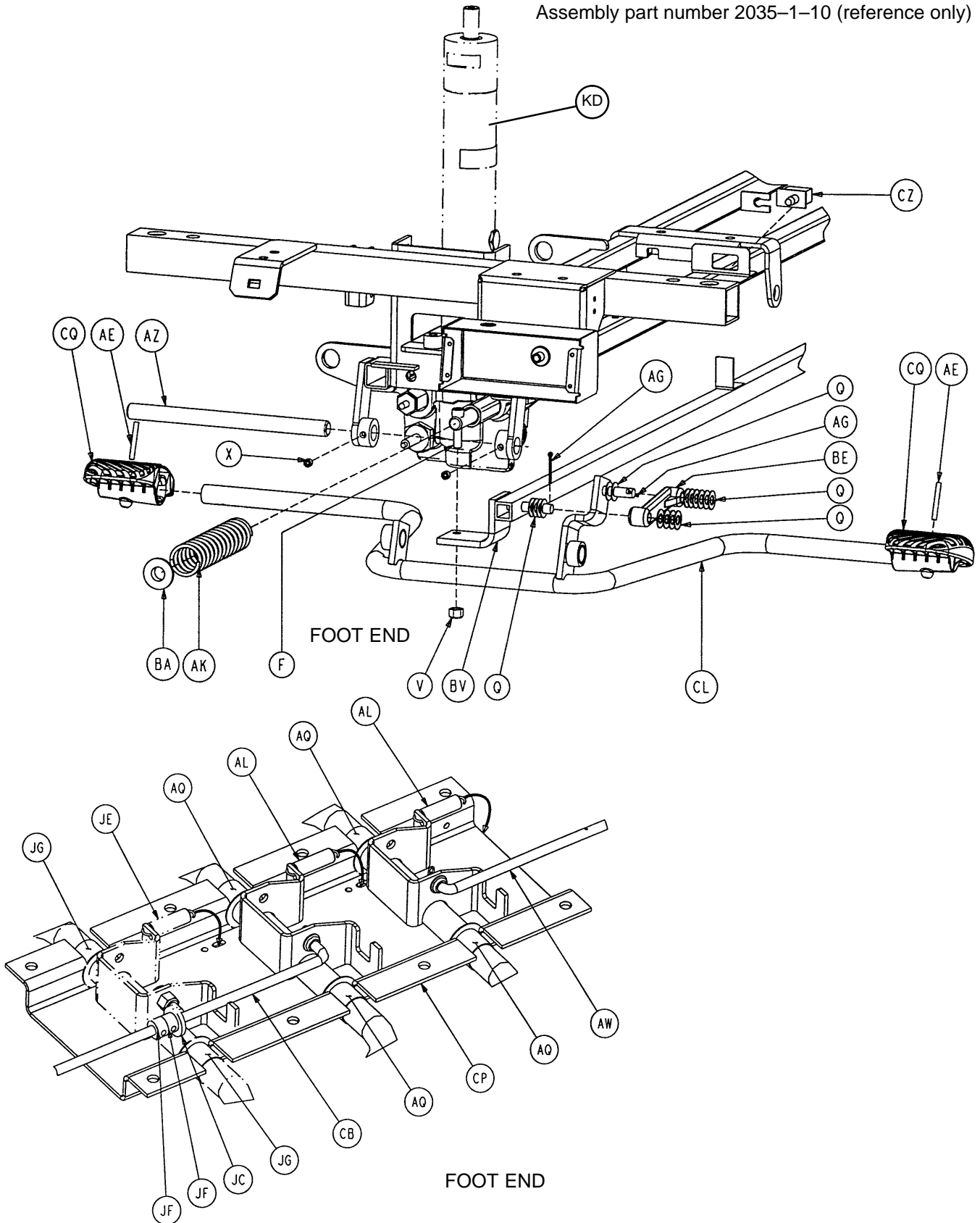



Base Assembly

Assembly part number 2035-1-10 (reference only)


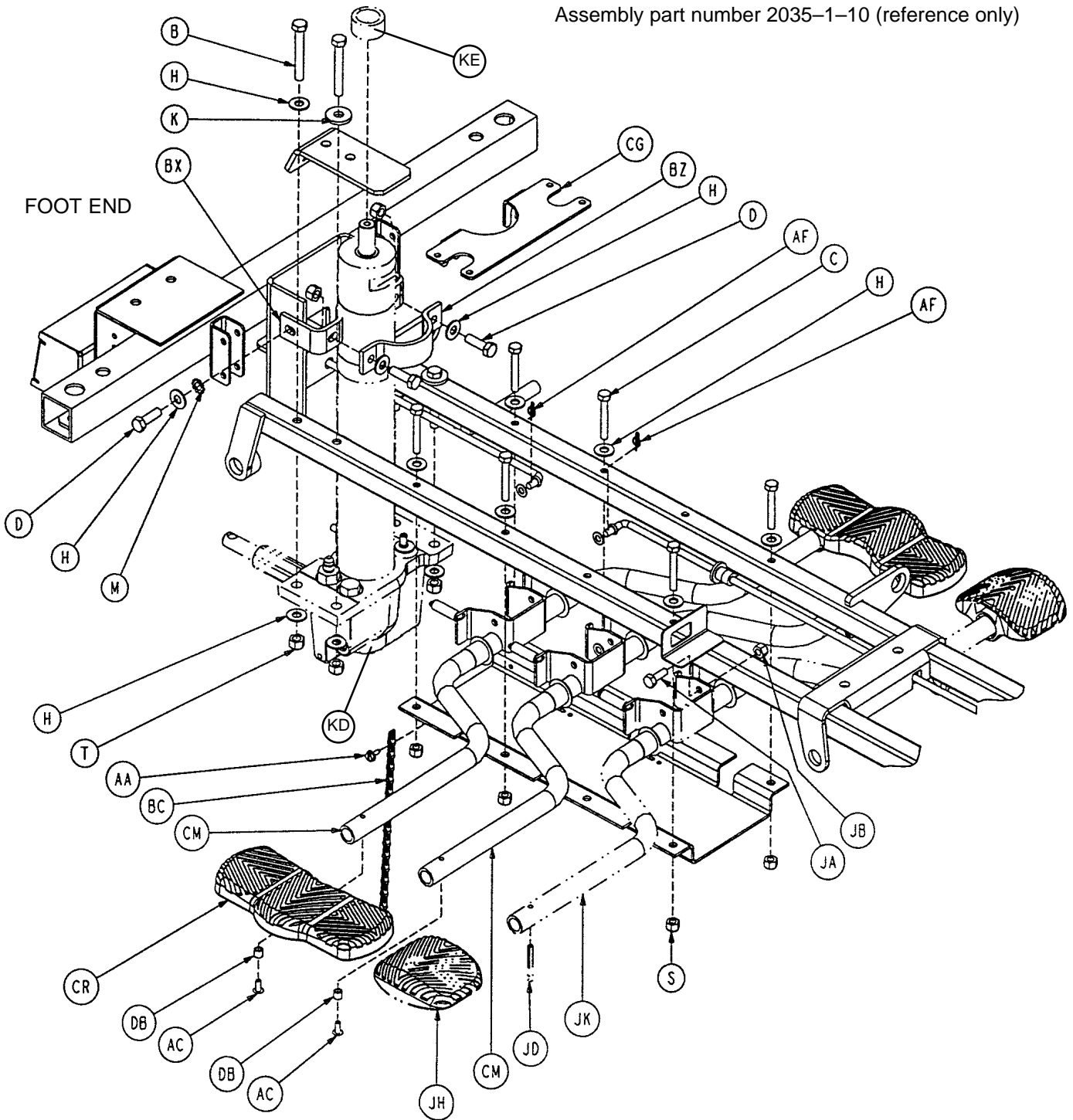


Base Assembly

Assembly part number 2035-1-10 (reference only)

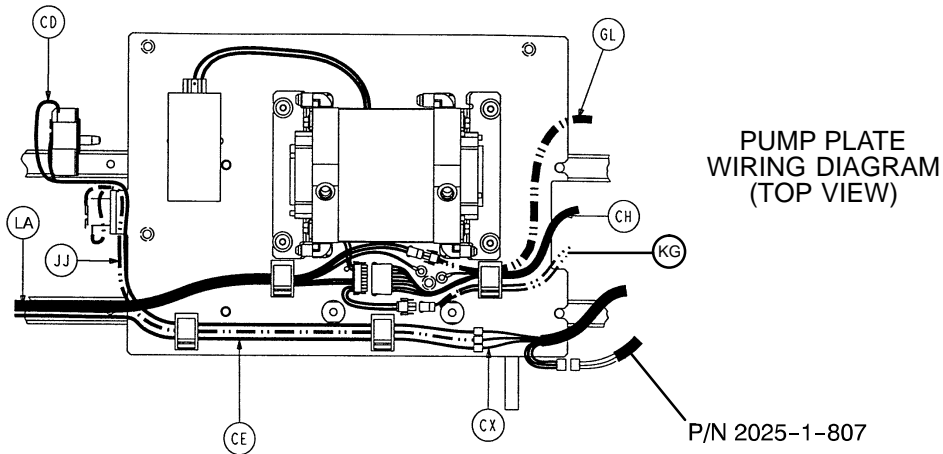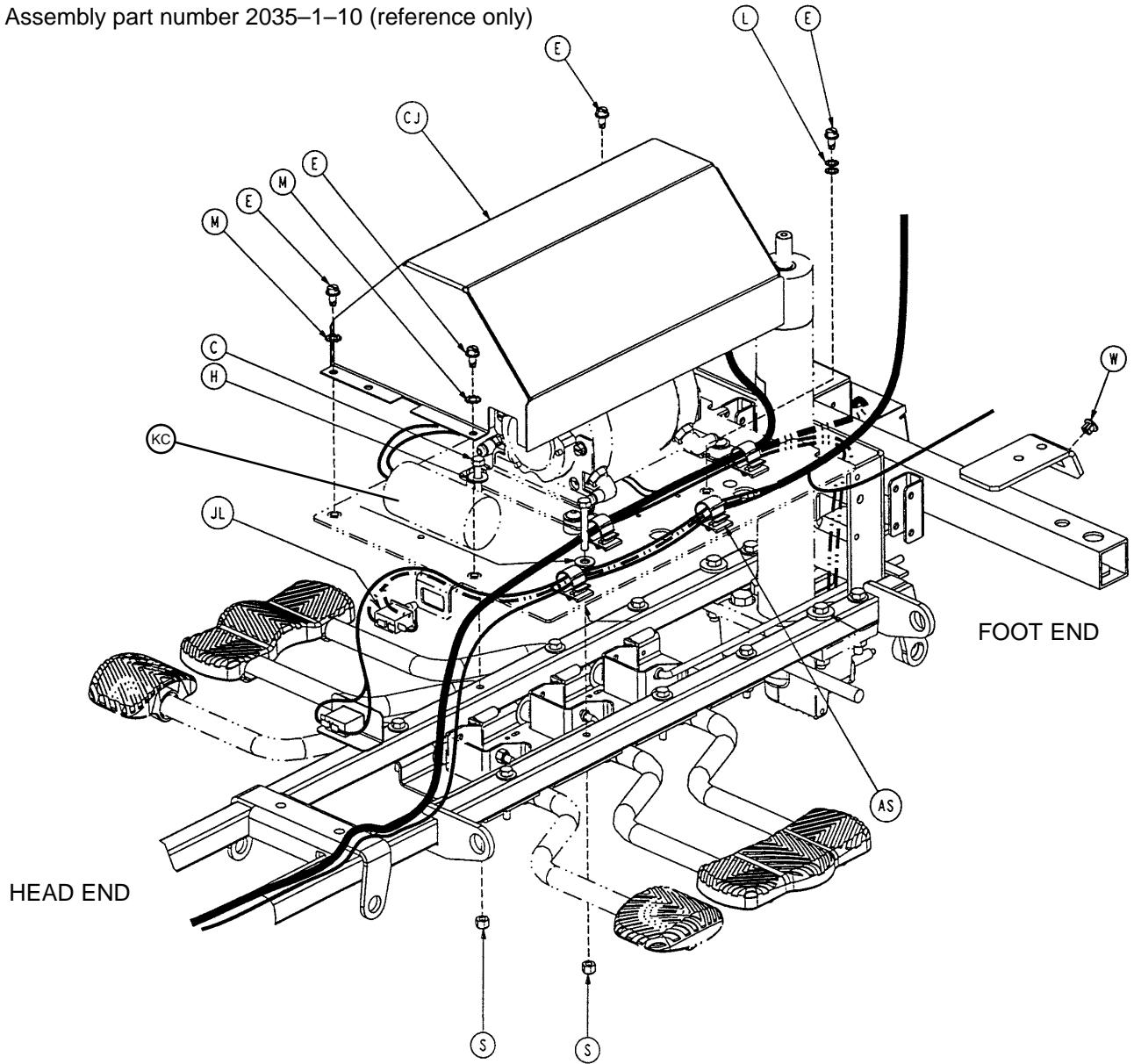


Base Assembly

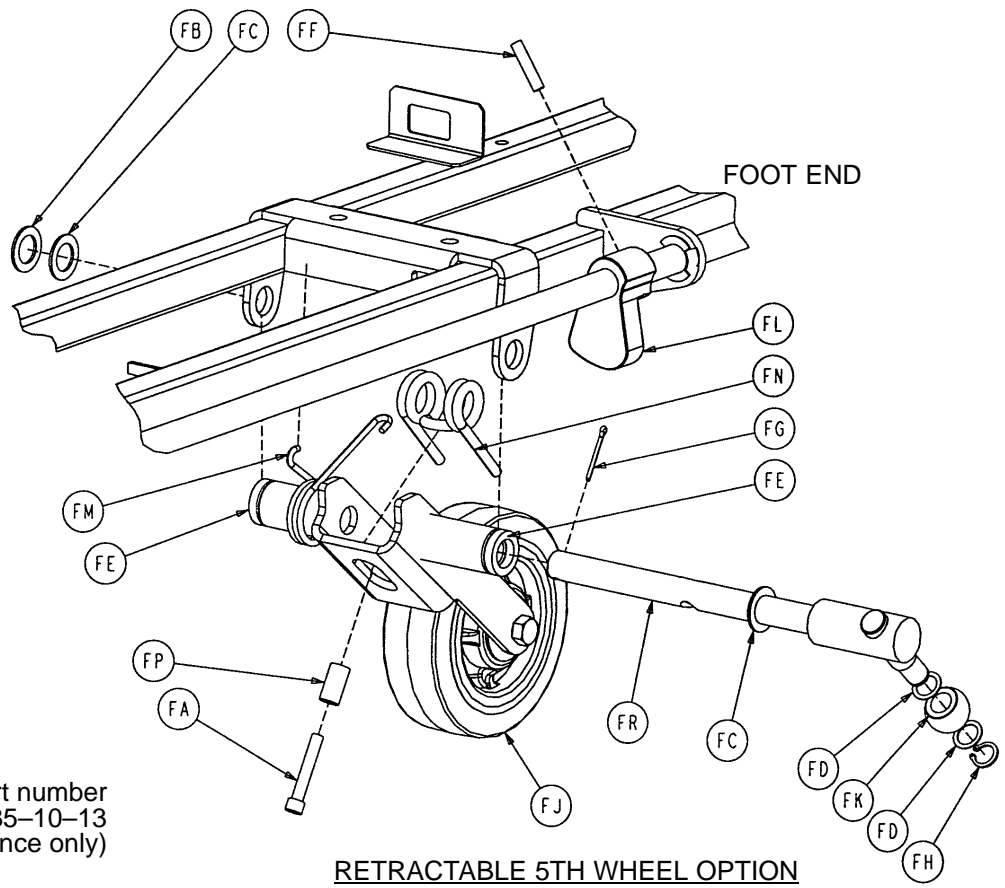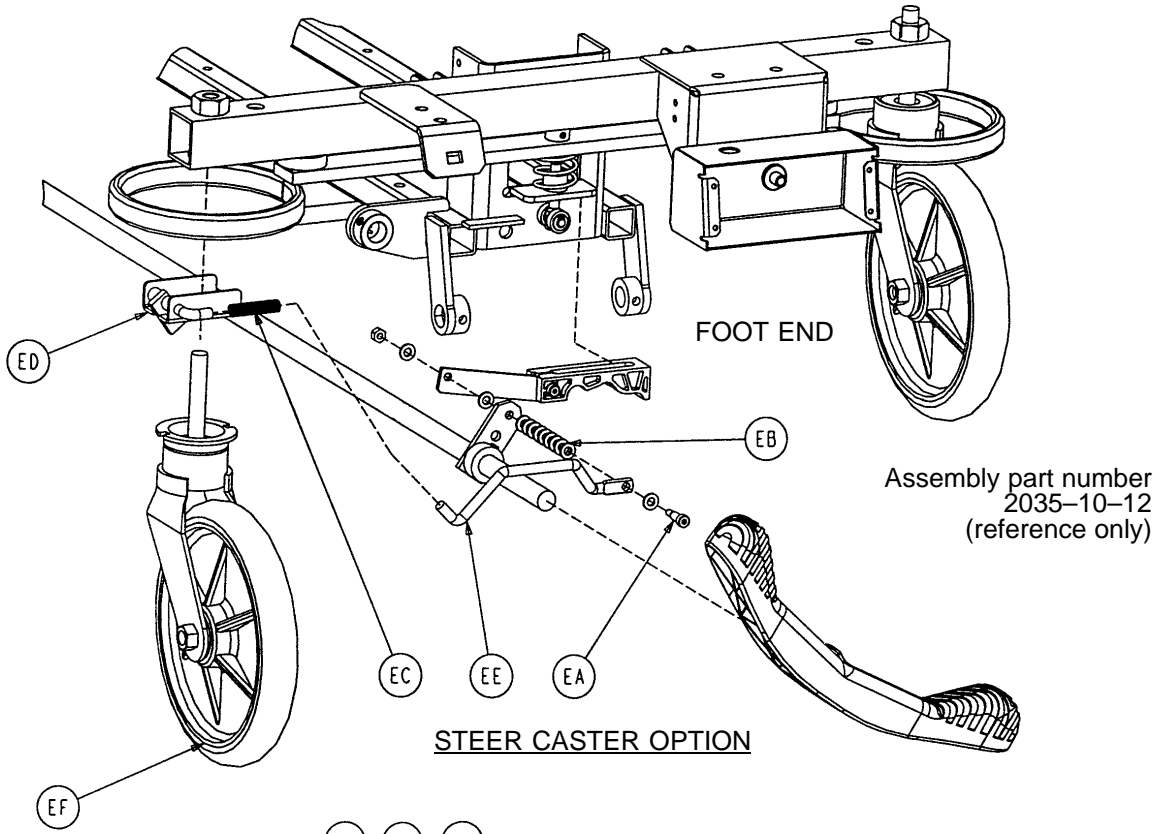


Base Assembly

NO 110V OUTLET OPTION
FOOT END

Assembly part number
2035-10-15
(reference only)

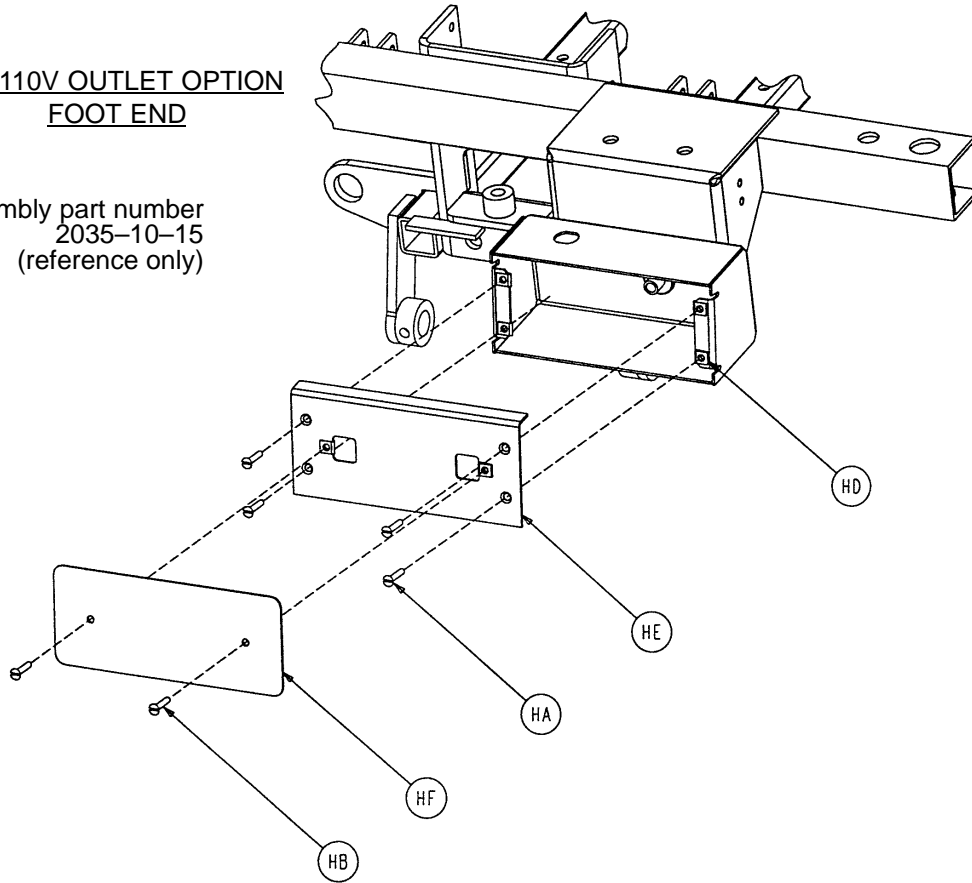


NO 110V OUTLET OPTION
HEAD END

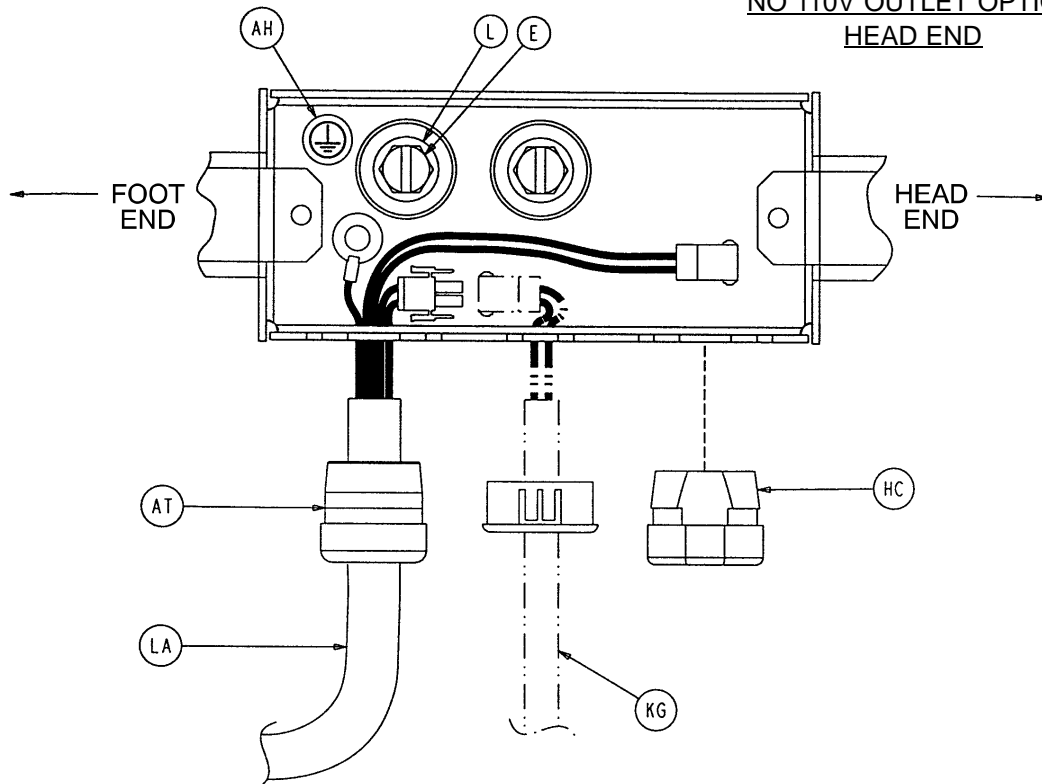


Base Assembly

110V OUTLET OPTION
FOOT END

Assembly part number
2035-10-14
(reference only)

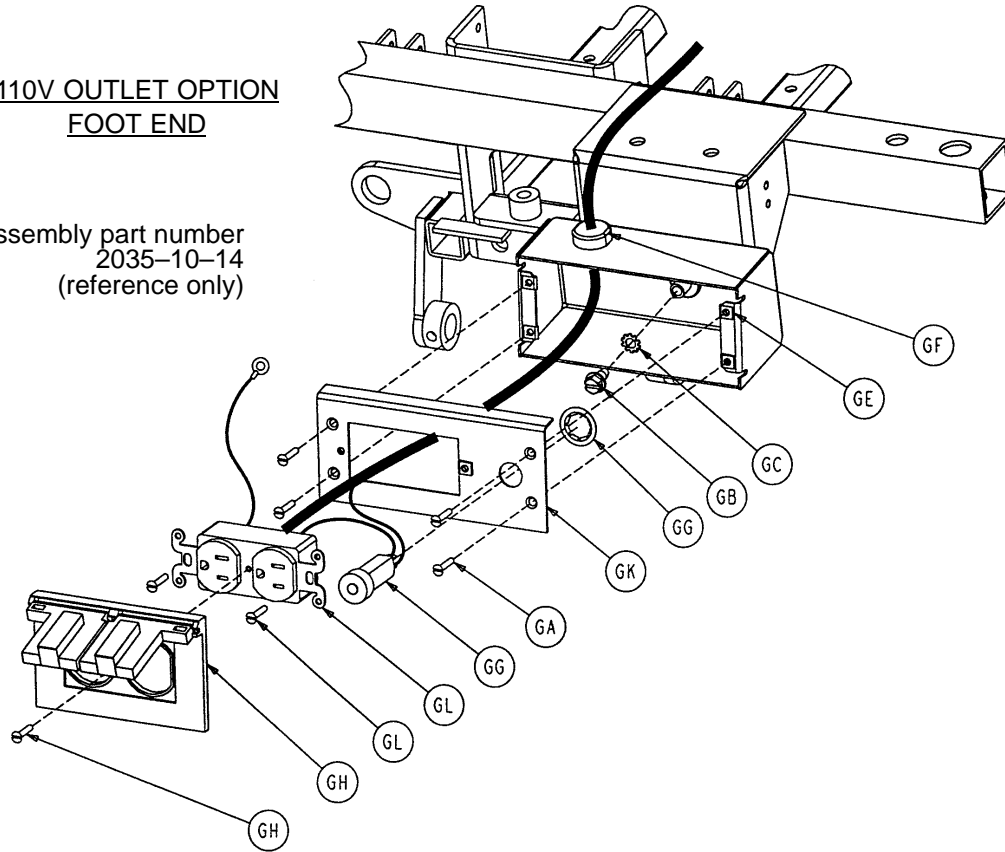


110V OUTLET OPTION
HEAD END

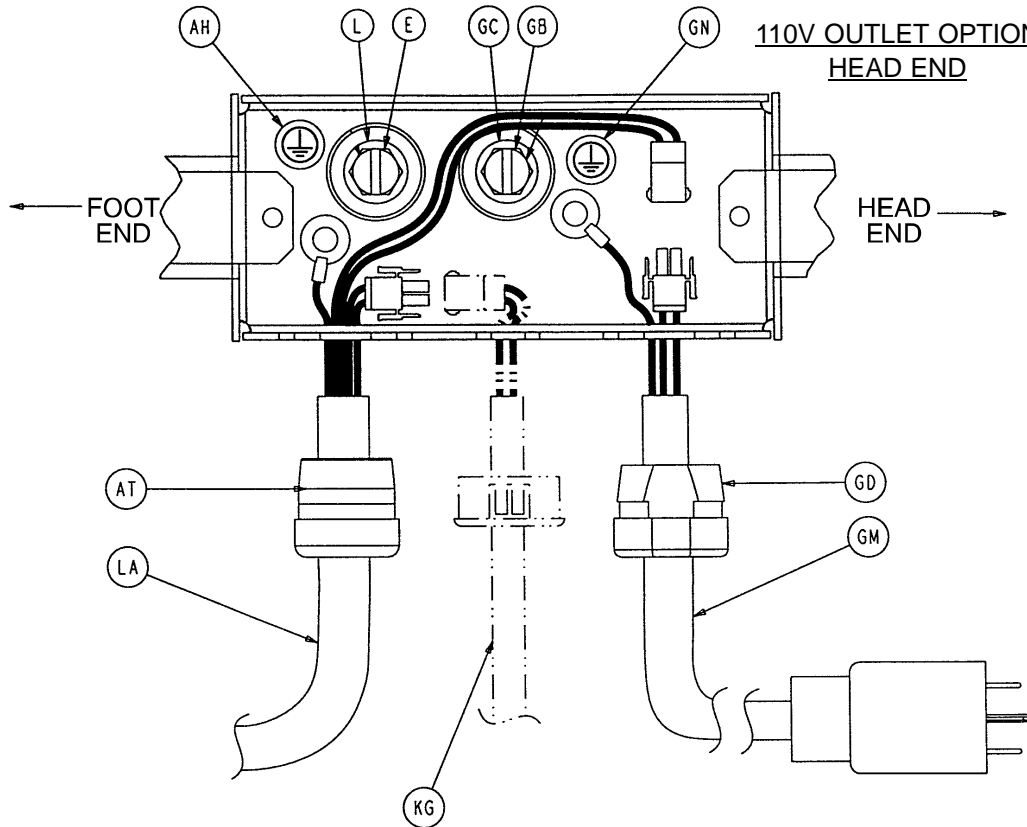

Base Assembly

DOMESTIC HOOD ASSEMBLY

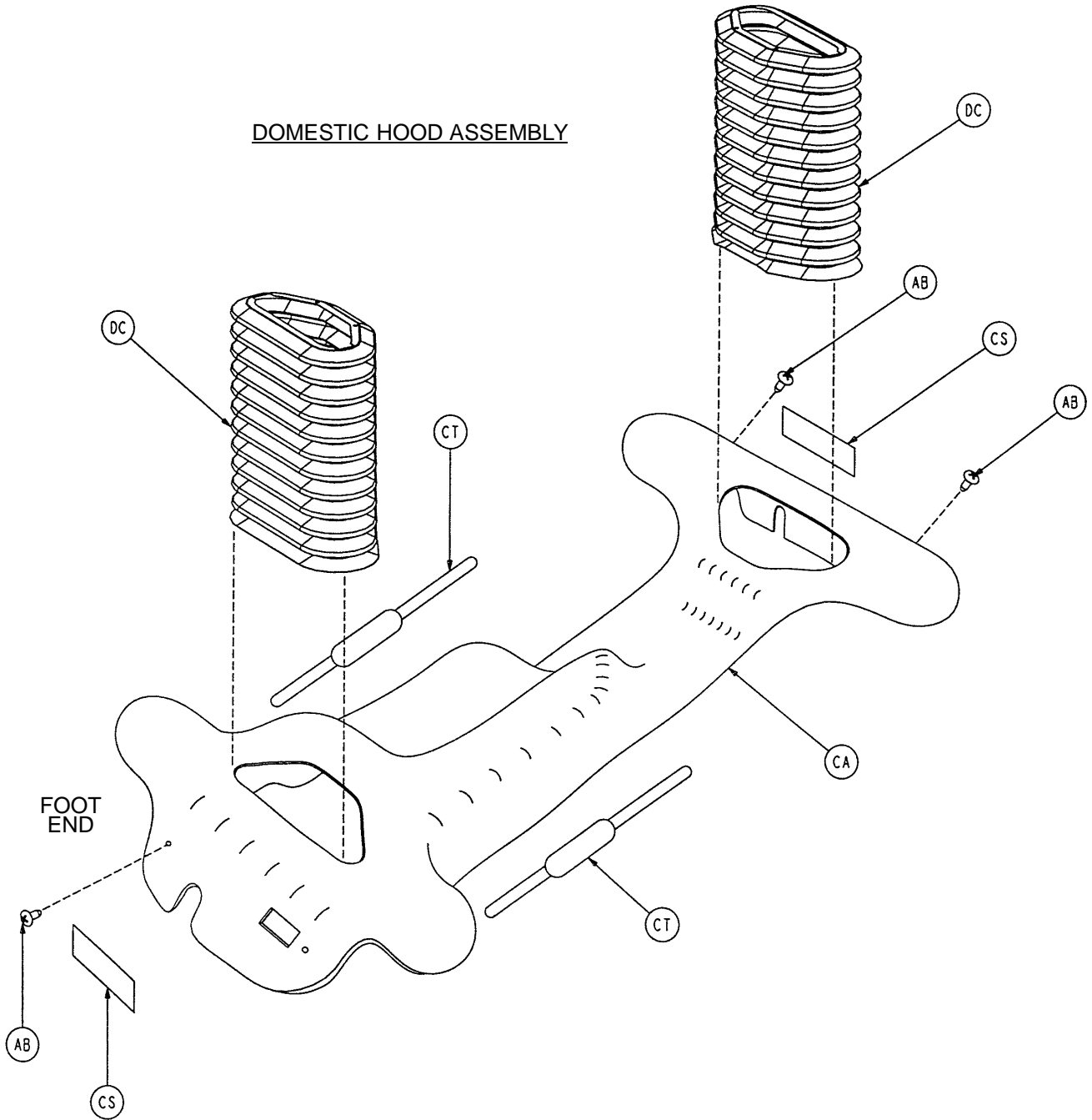

Base Assembly

Item	Part No.	Part Name	Qty.	Item	Part No.	Part Name	Qty.
A	1-87	Flat Hd. Slotted Screw	2	BC	715-1-156	Ground Chain	1
B	3-62	Hex Hd. Cap Screw	8	BD	715-1-158	Caster Nut	4
C	3-78	Hex Hd. Cap Screw	6	BE	715-1-214	Connecting Link	1
D	3-85	Hex Hd. Cap Screw	8	BF	715-1-333	Rel. Valve Stop Sleeve	2
E	3-221	Hex Washer Hd. Screw	5	BG	715-1-346	Release Paddle	2
F	4-146	Soc. Hd. Cap Screw	2	BH	715-201-94	Compression Spring	2
G	8-17	Shoulder Bolt	2	BJ	(page 11-15)	Brake Adjuster Ass'y	2
H	11-3	Washer	28	BK	(page 11-16)	Brake Cam Ass'y	2
K	11-262	Washer	4	BL	763-1-16	Spring Holder	1
L	13-10	Star Washer	3	BM	946-1-116	Brake Bar Bushing	4
M	13-38	Lock Washer	6	BN	1080-2-55	Actuator Plate Ass'y	2
N	14-2	Nylon Washer	6	BP	1210-201-251	Insert	4
P	14-4	Thrust Washer	4	BQ	1210-201-335	Brake Label	2
Q	14-20	Washer	12	BR	1210-201-336	Steer Label	2
R	16-2	Nylock Nut	2	BT	1550-90-13	Shock Caution Label	1
S	16-28	Nylock Nut	8	BV	2025-1-53	Pump Rod Connector	1
T	16-36	Nylock Hex Nut	16	BW	2025-1-58	Caster Brake Ring	1
V	16-48	Nylock Hex Nut	2	BX	2025-1-75	Outer Jack Support	2
W	18-36	Plastic Clip Nut	3	BZ	2025-1-76	Inner Jack Support	2
X	21-22	Set Screw	2	CA	2035-1-77	Base Hood	1
Z	21-50	Set Screw	4	CB	2025-1-79	Long Release Rod	1
AA	23-25	Hex Washer Hd. Screw	1	CC	2025-1-80	Pow. Cord Con. Box Cov.	1
AB	23-92	Phillips Truss Hd. Screw	3	CD	2025-1-804	Pump Switch Cable	1
AC	25-77	Rivet	4	CE	2025-1-805	Base Cable (DC)	1
AD	26-13	Roll Pin	2	CF	(page 11-17)	Caster Horn & Brg. Ass'y	3
AE	26-261	Groove Pin	4	CG	2025-201-96	Coil Cord Ret. Bracket	1
AF	27-20	Rue Ring	2	CH	2025-201-802	Lift Power Coil Cord (AC)	1
AG	27-7	Cotter Pin	2	CJ	2025-375-89	Pump/Motor Top Cover	1
AH	36-46	Ground Label	2	CK	2035-1-50	Base Frame Weldment	1
AJ	38-151	Cable Tie	14	CL	2035-1-57	Pump Pedal Weldment	1
AK	38-246	Compression Spring	1	CM	2035-1-59	Release Pedal Weldment	2
AL	38-355	Extension Spring	2	CN	2035-1-73	Brake Rod	1
AN	42-20	Collar	2	CP	2035-1-105	Release Pedal Bracket	1
AP	52-245	Nyliner Bearing	3	CQ	2035-1-126	Slip-On Pump Pedal	2
AQ	52-747	Flanged Bearing	4	CR	2035-1-127	Uni-Lowering Pedal	2
AR	52-783	U-Clip	2	CS	2035-1-128	Brake/Steer Label Set	1
AS	59-133	Push-Mount Wire Clip	4	CT	2035-1-130	Base Control Label Set	1
AT	59-169	Strain Relief	1	CV	2035-1-153	Butterfly "V" Pedal	2
AV	715-1-11	Brake Cushion	4	CX	2035-1-801	Lift Coil Cord (DC)	1
AW	715-1-46	Release Rod, Control End	1	CZ	3000-300-58	Cherry Switch	1
AX	715-1-61	Caster Brake Weldment	1	DA	3000-300-604	Warning Label	1
AZ	715-1-92	Pump Pedal Shaft	1	DB	715-1-333	Spacer	4
BA	715-1-133	Collar	1	DC	2030-000-101	Bellows	2
BB	715-1-140	Vinyl Tubing	1				

Base Assembly

Steer Caster Option

Item	Part No.	Part Name	Qty.
EA	8-30	Soc. Hd. Shoulder Bolt	1
EB	14-2	Nylon Washer	5
EC	38-211	Compression Spring	1
ED	1000-59-10	Latch Assembly	1
EE	1000-210-62	Lock Linkage Bar	1
EF	(page 11-18)	Steer Caster Assembly	1

Retractable 5th Wheel Option

Item	Part No.	Part Name	Qty.
FA	4-10	Soc. Hd. Shoulder Bolt	1
FB	11-410	Washer	1
FC	14-40	Nylon Washer	2
FD	14-79	Nylon Washer	2
FE	52-309	Spacer	2
FF	26-13	Roll Pin	1
FG	27-16	Cotter Pin	1
FH	28-131	Retaining Pin	1
FJ	(page 11-20)	Wheel Arm Assembly	1
FK	1210-1-144	Roller	1
FL	1210-1-145	Cam	1
FM	12101-1-146	Torsion Spring	1
FN	1210-1-148	Torsion Spring	1
FP	1210-201-149	Spring Spacer	1
FQ	(page 11-17)	Caster Horn & Brg. Ass'y	1
FR	2035-10-26	Pivot Rod Weldment	1

110V Outlet Option

Item	Part No.	Part Name	Qty.
GA	1-87	Flat Soc. Hd. Screw	4
GB	3-221	Hex Washer Hd. Screw	2
GC	13-10	Star Washer	2
GD	30-27	Grommet	1
GE	52-783	U-Clip	6
GF	59-107	Strain Relief	1
GG	59-180	Circuit Breaker	1
GH	2020-11-222	Outlet Cover	1
GK	2035-10-20	110V Box Cover	1
GL	2035-10-801	110V Outlet Cable Ass'y	1
GM	2035-10-802	110V Option Power Cord	1
GN	36-46	Ground Label	2

No 110V Outlet Option

Item	Part No.	Part Name	Qty.
HA	1-87	Flat Soc. Hd. Screw	4
HB	4-126	Hex Soc. But. Hd. Screw	2
HC	37-208	Round Cap Plug	1
HD	52-783	U-Clip	6
HE	2025-10-22	Box Cover	1
HF	2025-10-801	Box Cover Plate	1

Instant Trend. Pedal Option

Item	Part No.	Part Name	Qty.
JA	3-50	Hex Hd. Cap Screw	1
JB	16-6	Lock Nut	1
JC	11-154	Washer	1
JD	26-261	Groove Pin	2
JE	38-355	Extension Spring	1
JF	42-13	Collar	2
JG	52-747	Flanged Bearing	2
JH	2035-1-126	Slip On Pump Pedal	2
JJ	2025-1-803	Power Trend Cable	1
JK	2035-1-59	Release Pedal Weldment	1
JL	3000-300-58	Switch Plunger	1

Base Assembly

2025–71 Low Ht. Hydraulic Ass'y (w/Solenoid)

Item	Part No.	Part Name	Qty.
KA	16–3	Nylock Nut	2
KB	30–38	Split Bushing	1
KC	59–140	Capacitor	1
KD	(page 11–22)	Jack/Motor Ass'y	1
KE	2025–71–99	Low Height Spacer	2
KF	2025–76–99	Solenoid Guard Bracket	1
KG	2025–276–801	Solenoid Cable Ass'y	2

2036–71 Int. Low Ht. Hyd. Ass'y (w/Solenoid)

Item	Part No.	Part Name	Qty.
KA	16–3	Nylock Nut	2
KB	30–38	Split Bushing	1
KC	59–156	Capacitor	1
KD	(page 11–22)	Jack/Motor Ass'y	1
KE	2025–71–99	Low Height Spacer	2
KF	2025–76–99	Solenoid Guard Bracket	1
KG	2036–76–801	Solenoid Cable Ass'y	2

2025–76 Std. Ht. Hydraulic Ass'y (w/Solenoid)

Item	Part No.	Part Name	Qty.
KA	16–3	Nylock Nut	2
KB	30–38	Split Bushing	1
KC	59–140	Capacitor	1
KD	(page 11–22)	Jack/Motor Ass'y	1
KF	2025–76–99	Solenoid Guard Bracket	1
KG	2025–276–801	Solenoid Cable Ass'y	2

2036–76 Int. Std. Ht. Hyd. Ass'y (w/Solenoid)

Item	Part No.	Part Name	Qty.
KA	16–3	Nylock Nut	2
KB	30–38	Split Bushing	1
KC	59–156	Capacitor	1
KD	(page 11–22)	Jack/Motor Ass'y	1
KF	2025–76–99	Solenoid Guard Bracket	1
KG	2036–76–801	Solenoid Cable Ass'y	2


Domestic Base Assembly

Item	Part No.	Part Name	Qty.
LA	2035–1–806	Base Cable (AC)	1

International Base Assembly


Item	Part No.	Part Name	Qty.
LA	2036–1–806	230V Base Cable (AC)	1

715-201-150 Brake Adjuster Assembly

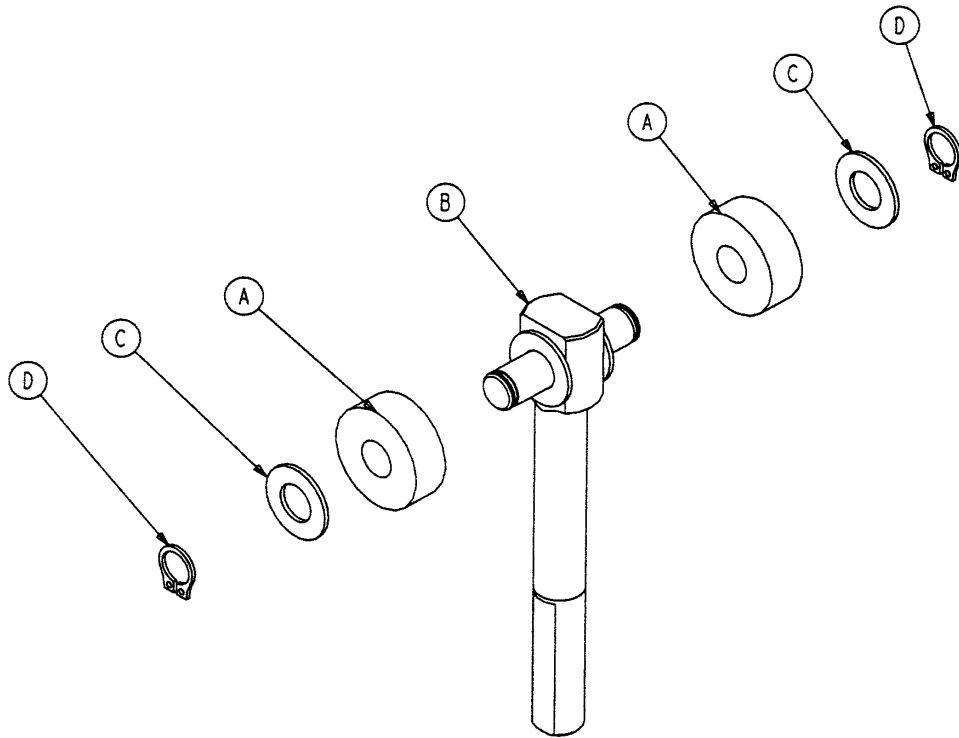


Item	Part No.	Part Name	Qty.
A	715-1-180	Cam Bearing	2
B	715-201-62	Threaded Stud Assembly	1
C	14-4	Nylon Washer	4
D	28-8	External Retaining Ring	2

715–201–213 Brake Cam Assembly

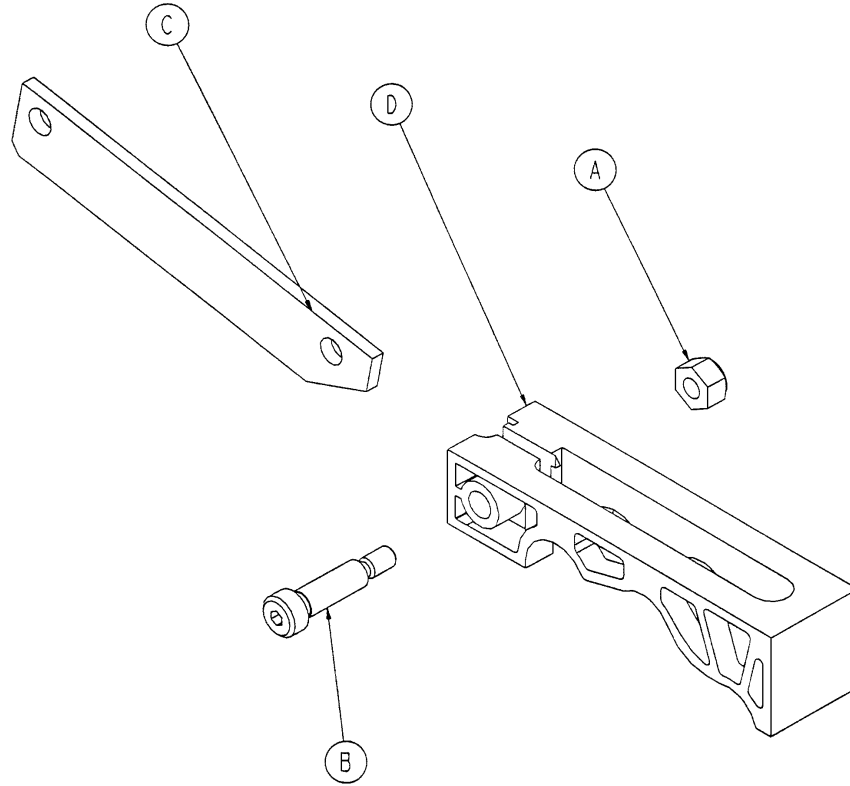


Item	Part No.	Part Name	Qty.
A	16–59	Hex Nut	1
B	8–21	Soc. Hd. Shoulder Screw	1
C	715–201–173	Brake Connecting Link	1
D	715–301–221	Brake Cam	1

2025–10–25 Caster Assembly

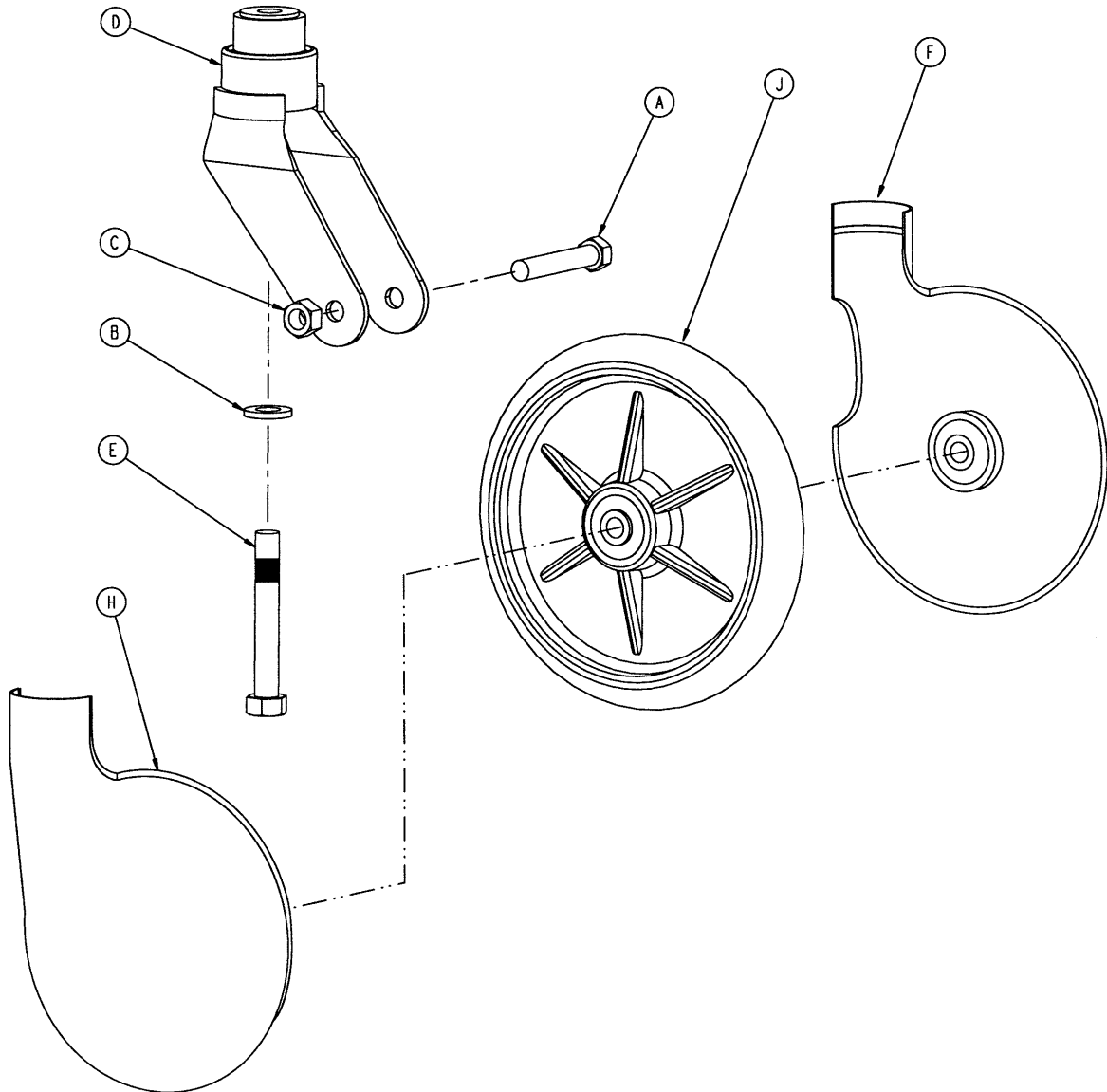


Item	Part No.	Part Name	Qty.
A	3-99	Hex Hd. Cap Screw	1
B	11-310	Washer	1
C	16-60	Hex Nut	1
D	715-2-16	Horn Assembly	1
E	715-3-96	Hex Hd. Cap Screw	1
F	2025-1-47	Wheel Cover, Right	1
H	2025-1-48	Wheel Cover, Left	1
J	(page 11-19)	Molded Wheel Assembly	1

2025-10-24 Steer Caster Assembly

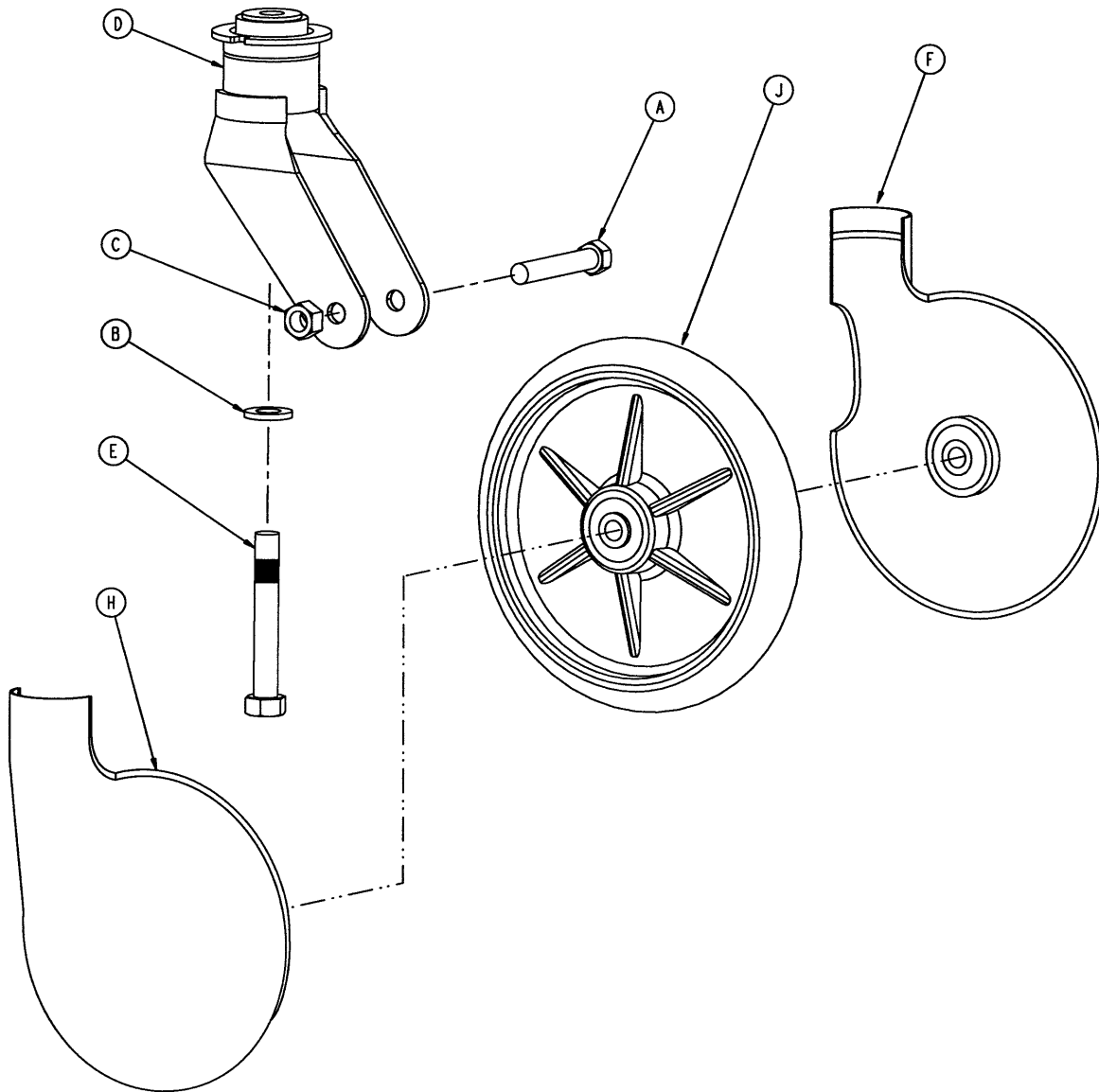


Item	Part No.	Part Name	Qty.
A	3-99	Hex Hd. Cap Screw	1
B	11-310	Washer	1
C	16-60	Hex Nut	1
D	700-10-50	Steering Horn Assembly	1
E	715-3-96	Hex Hd. Cap Screw	1
F	2025-1-47	Wheel Cover, Right	1
H	2025-1-48	Wheel Cover, Left	1
J	(page 11-19)	Molded Wheel Assembly	1

715-2-25 Wheel Assembly

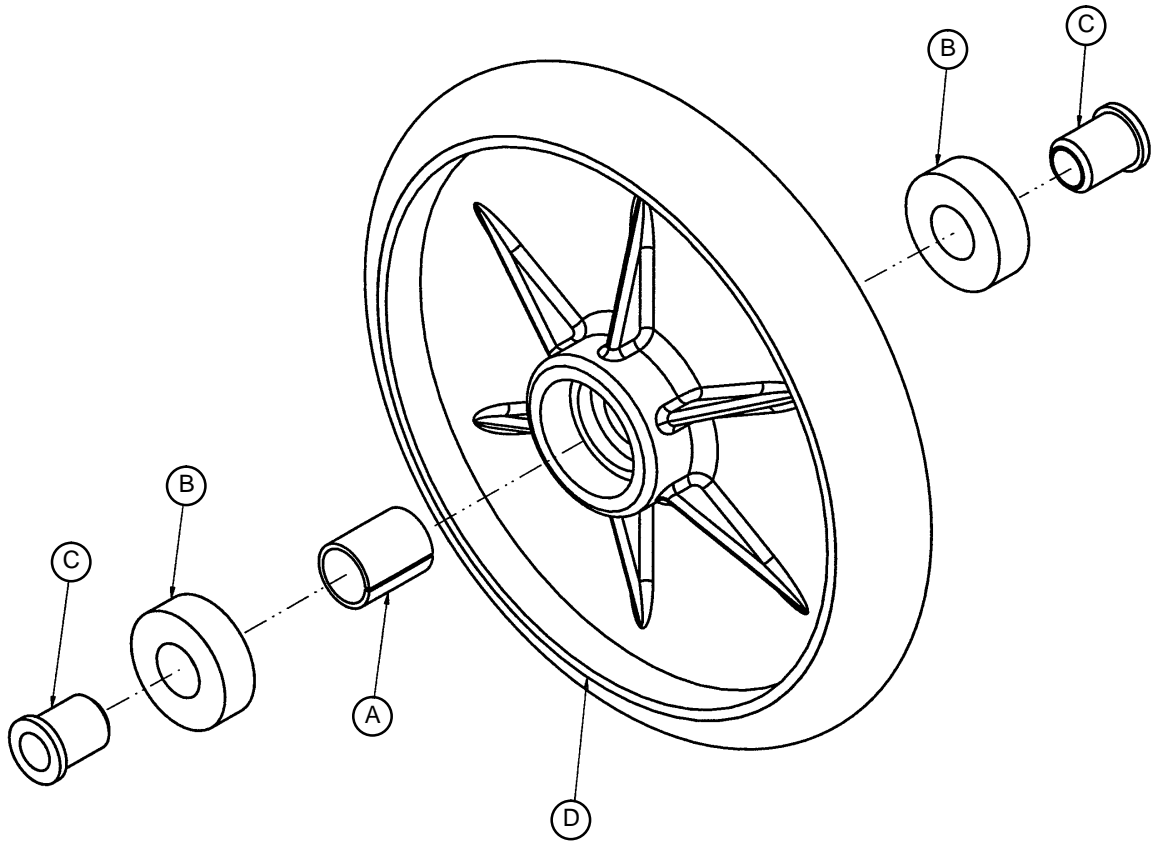

Item	Part No.	Part Name	Qty.
A	52-503	Bearing Spacer	1
B	81-226	Bearing	2
C	715-1-255	Wheel Bearing	2
D	715-2-124	Wheel	1

715–201–125 Fifth Wheel Arm Assembly

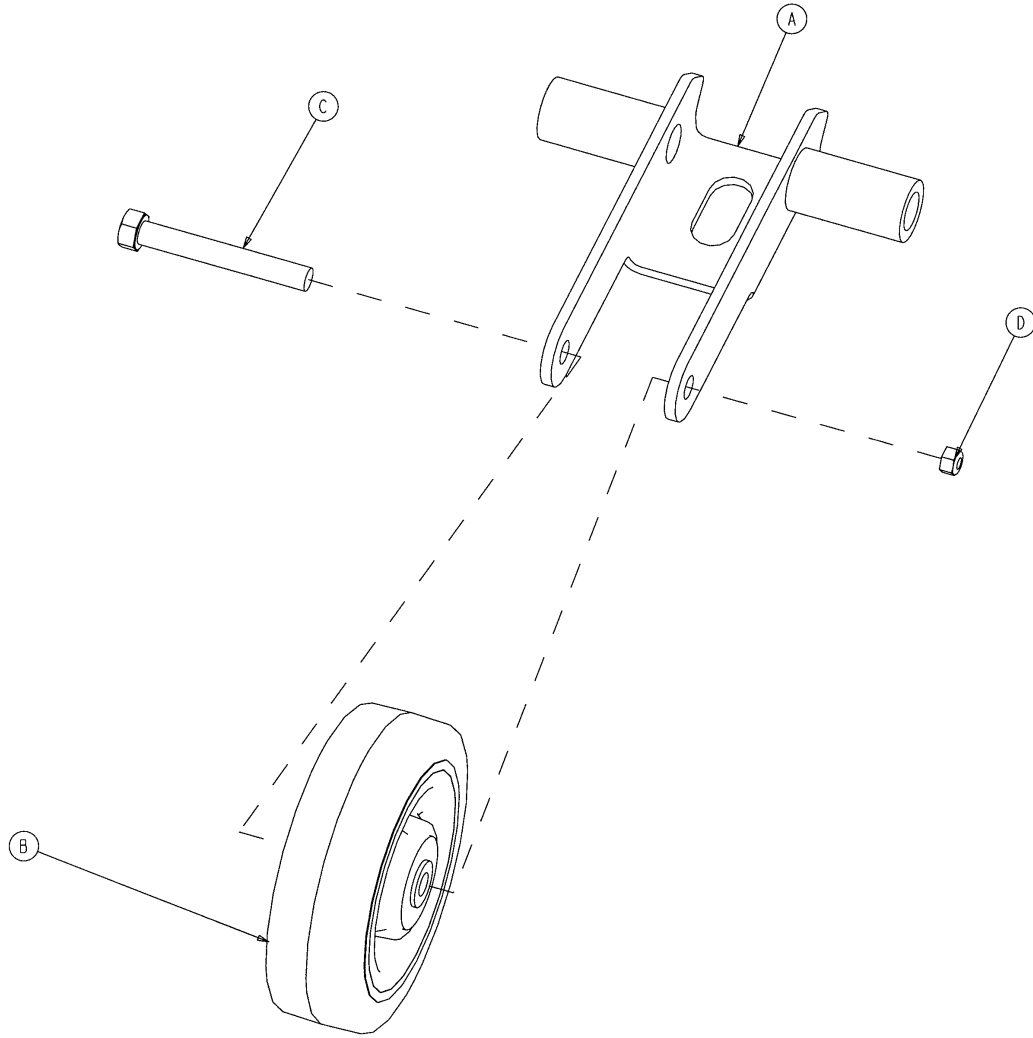



Item	Part No.	Part Name	Qty.
A	1210–201–553	Wheel Arm	1
B	1210–1–147	Fifth Wheel	1
C	3–31	Hex Hd. Cap Screw	1
D	16–12	Nylock Nut	1

Notes


Standard and Low Height Jack/Motor Assembly

Assembly part numbers:


- 2025-271-10 Low Height Jack/Motor
- 2025-276-10 Standard Jack/Motor
- 2026-271-10 International Low Height Jack/Motor
- 2026-276-10 International Standard Jack Motor

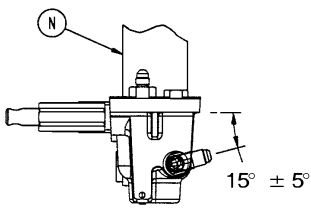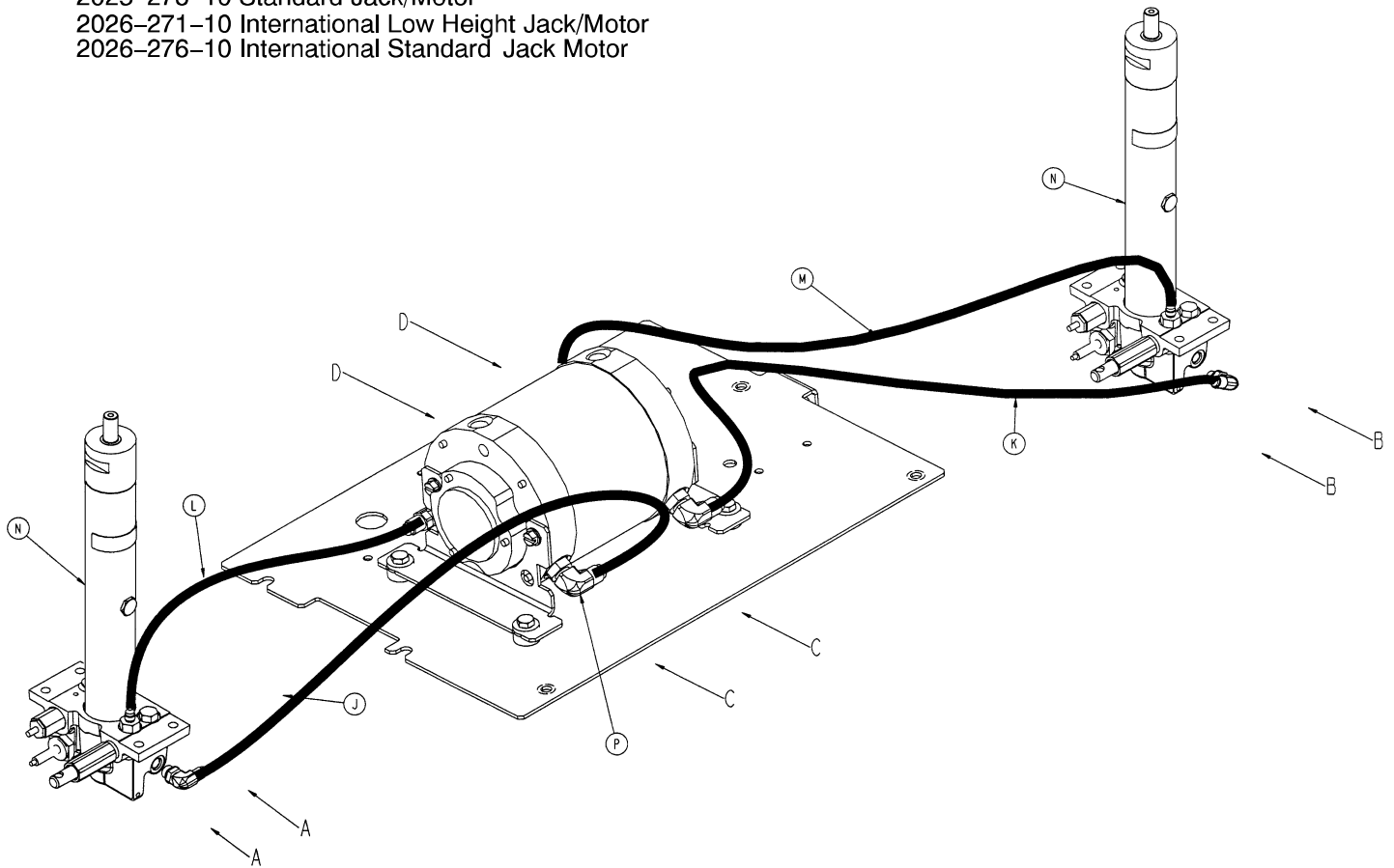


VIEW A-A

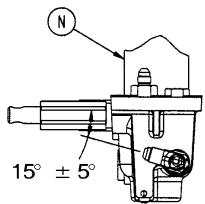

VIEW B-B

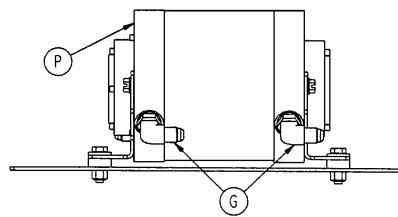

VIEW C-C

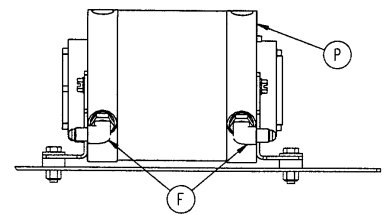


VIEW D-D

Standard and Low Height Jack/Motor Assembly


Assembly part numbers:

- 2025-271-10 Low Height Jack/Motor
- 2025-276-10 Standard Jack/Motor
- 2026-271-10 International Low Height Jack/Motor
- 2026-276-10 International Standard Jack Motor

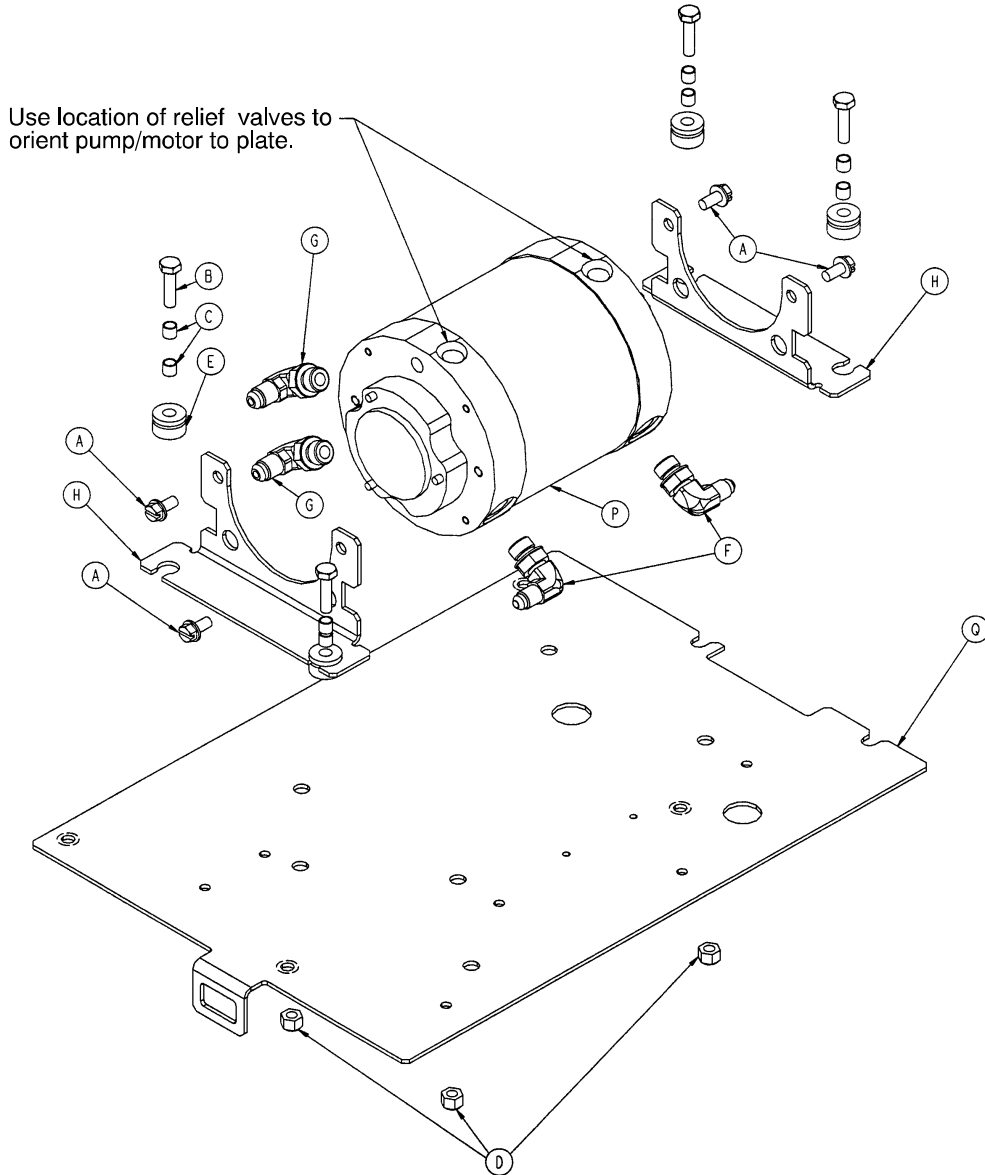

Item	Part No.	Part Name	Qty.	Item	Part No.	Part Name	Qty.
A	3-121	Hex Wash. Hd. Screw	4	J	2025-75-91	Short Return Line	1
B	3-3	Hex Hd. Cap Screw	4	K	2025-275-92	Long Return Line	1
C	715-1-333	Rel. Valve Stop Sleeve	8	L	2025-75-93	Short Pressure Line	1
D	16-28	Nylock Nut	4	M	2025-75-94	Long Pressure Line	1
E	30-40	E-A-R Grommet	4	N	(page 11-24)	Jack Assembly	2
F	48-144	90° Elbow O-Ring Fitting	2	P	2025-275-16	Pump/Motor	1
G	48-155	90° Elbow O-Ring Fitting	4		2026-275-16	International Pump/Motor	1
H	2025-75-84	Mounting Bracket	1	Q	2025-375-90	Bottom Plate	1
				R	3000-300-402	2-Sided Tape	1

Standard and Low Height Jack

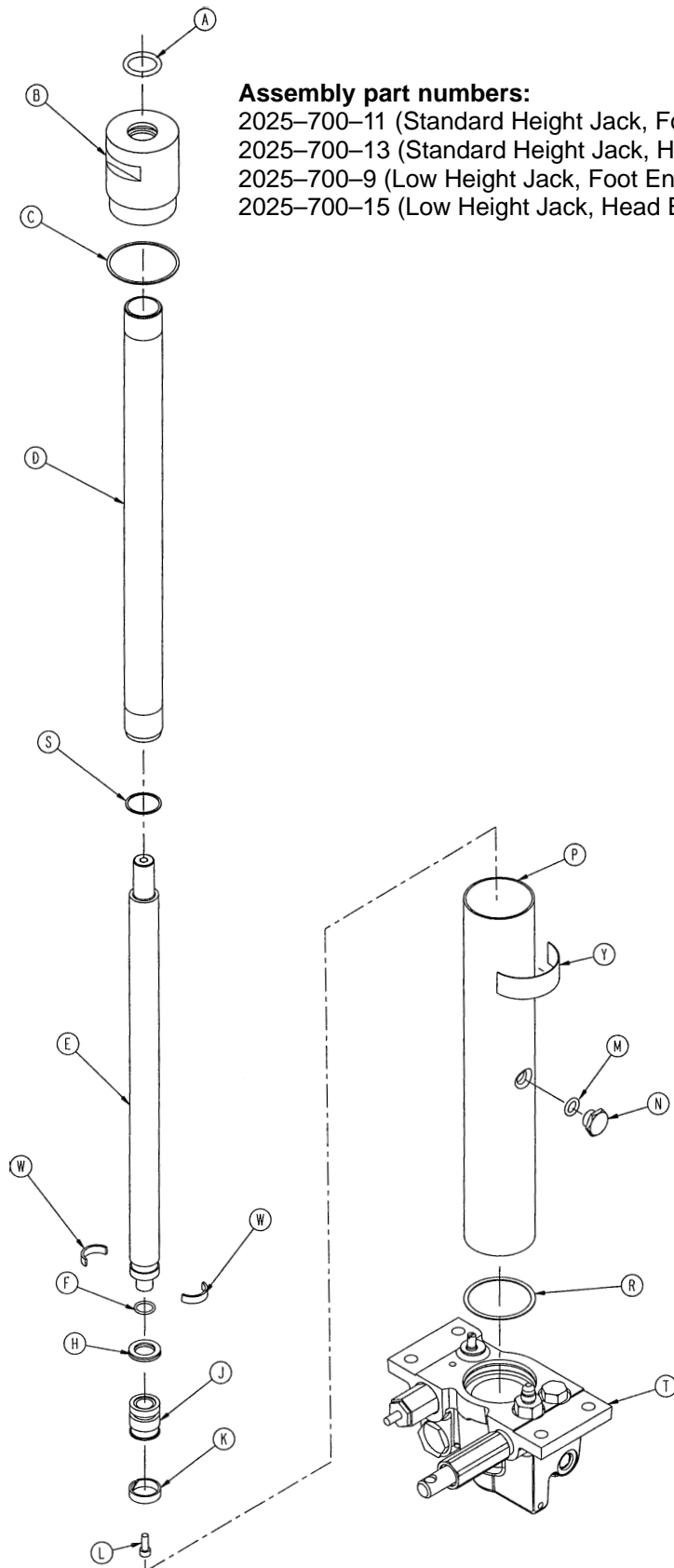


Standard and Low Height Jack

Jack Assembly				Low Height Jack Assembly			
Item	Part No.	Part Name	Qty.	Item	Part No.	Part Name	Qty.
A	45-904	Quad Ring	1	A	45-904	Quad Ring	1
B	1550-70-6	Cap Assembly	1	B	1550-70-6	Cap Assembly	1
C	390-1-243	Gasket	1	C	390-1-243	Gasket	1
D	1550-70-4	Actuator Cylinder	1	D	2025-70-98	Actuator Cylinder, Low Ht.	1
E	2025-270-95	Actuator	1	E	2025-270-99	Actuator, Low Height	1
F	45-14	O-Ring	1	F	45-14	O-Ring	1
H	926-20-161	Parker Packing	1	H	926-20-161	Parker Packing	1
J	715-1-331	Piston End	1	J	715-1-331	Piston End	1
K	926-20-162	Wear Ring	1	K	926-20-162	Wear Ring	1
L	4-14	Soc. Hd. Cap Screw	1	L	4-14	Soc. Hd. Cap Screw	1
M	45-110	O-Ring	1	M	45-110	O-Ring	1
N	388-1-38	Plug	1	N	388-1-38	Plug	1
P	2025-75-97	Reservoir	1	P	2025-70-97	Reservoir, Low Height	1
R	390-1-244	Gasket	1	R	390-1-244	Gasket	1
S	390-1-238	Actuator Gasket	1	S	390-1-238	Acturtor Gasket	1
T	(page 11-26)	Jack Base Assembly	1	T	(page 11-26)	Jack Base Assembly	1
W	390-2-139	Retaining Collar	2	W	390-2-139	Retaining Collar	2
Y	2025-276-96	Label	1	Y	2025-271-96	Low Height Label	1


Jack Base Assembly

Assembly part number 2025-76-14 (reference only)

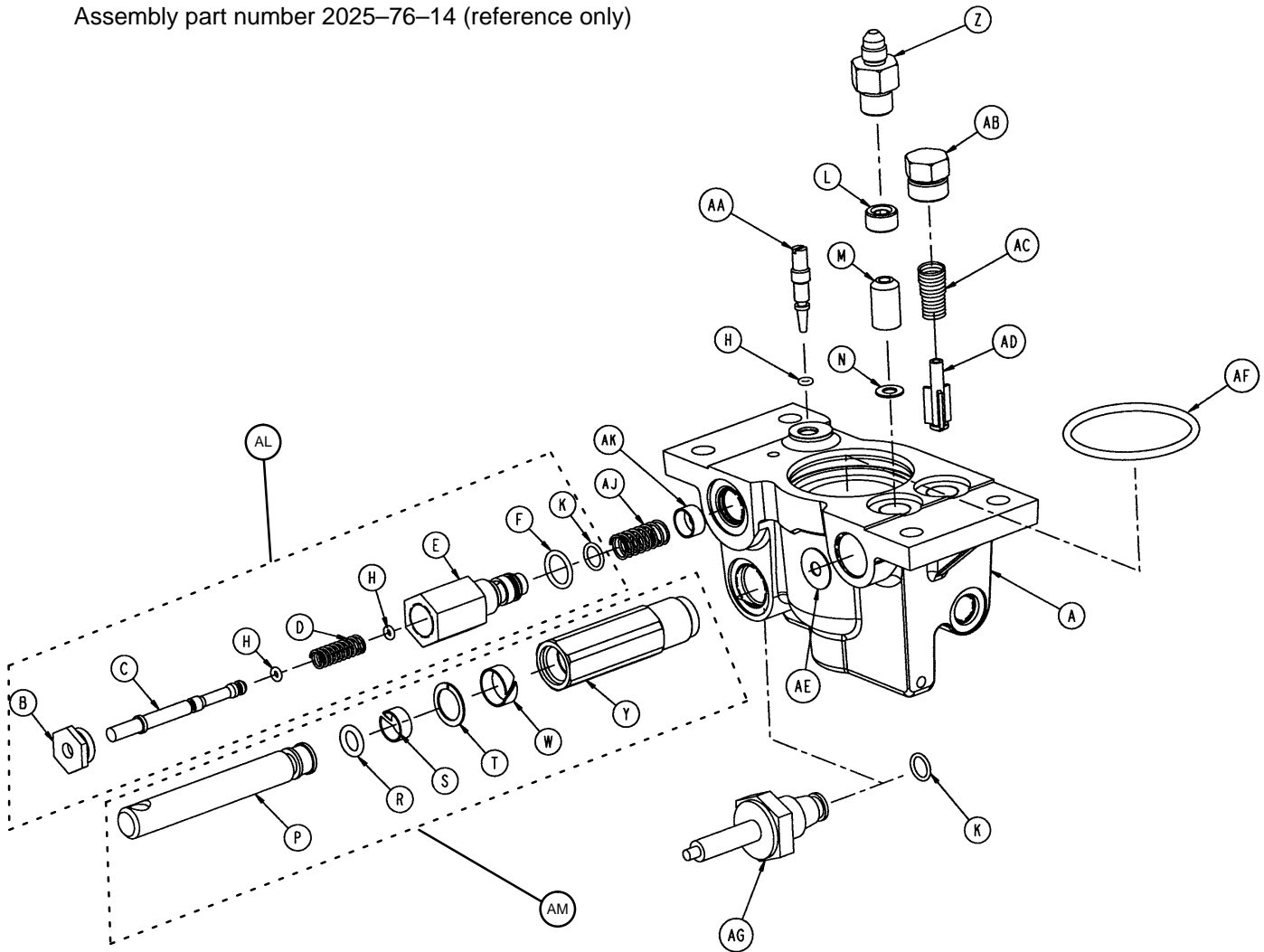


Item	Part No.	Part Name	Qty.	Item	Part No.	Part Name	Qty.
A	2025-75-98	Jack Base	1	T	14-50	Bearing Retainer	1
B	1210-70-13	Base Plug	1	W	715-1-327	Cylinder Wear Ring	1
C	715-270-1	Pin	1	Y	715-1-316	Pump Cylinder	1
D	38-311	Compression Spring	1	Z	48-150	Check Valve	1
E	2025-75-87	Pin Housing	1	AA	715-1-307	Needle Valve	1
F	45-967	O-Ring	1	AB	48-147	Hex Hd. O-Ring Plug	1
H	45-6	O-Ring	3	AC	390-2-134	Conical Comp. Spring	1
K	45-966	O-Ring	2	AD	715-1-341	Poppet	1
L	715-1-309	Valve Plug	1	AE	715-1-329	Pump Seal	1
M	926-20-153	Check Valve	1	AF	45-978	O-Ring	1
N	926-20-154	Seal	1	AG	2025-76-96	Solenoid Valve Body	1
P	715-1-318	Pump Piston	1	AJ	38-383	Compression Spring	1
R	45-110	O-Ring	1	AK	2025-75-85	Filter Screen	1
S	715-1-328	Piston Wear Ring	1	AL	2025-700-26	Release Valve Kit	1
				AM	715-100-325	Pump Piston Kit	1

715–100–325 Jack Pump Piston Assembly

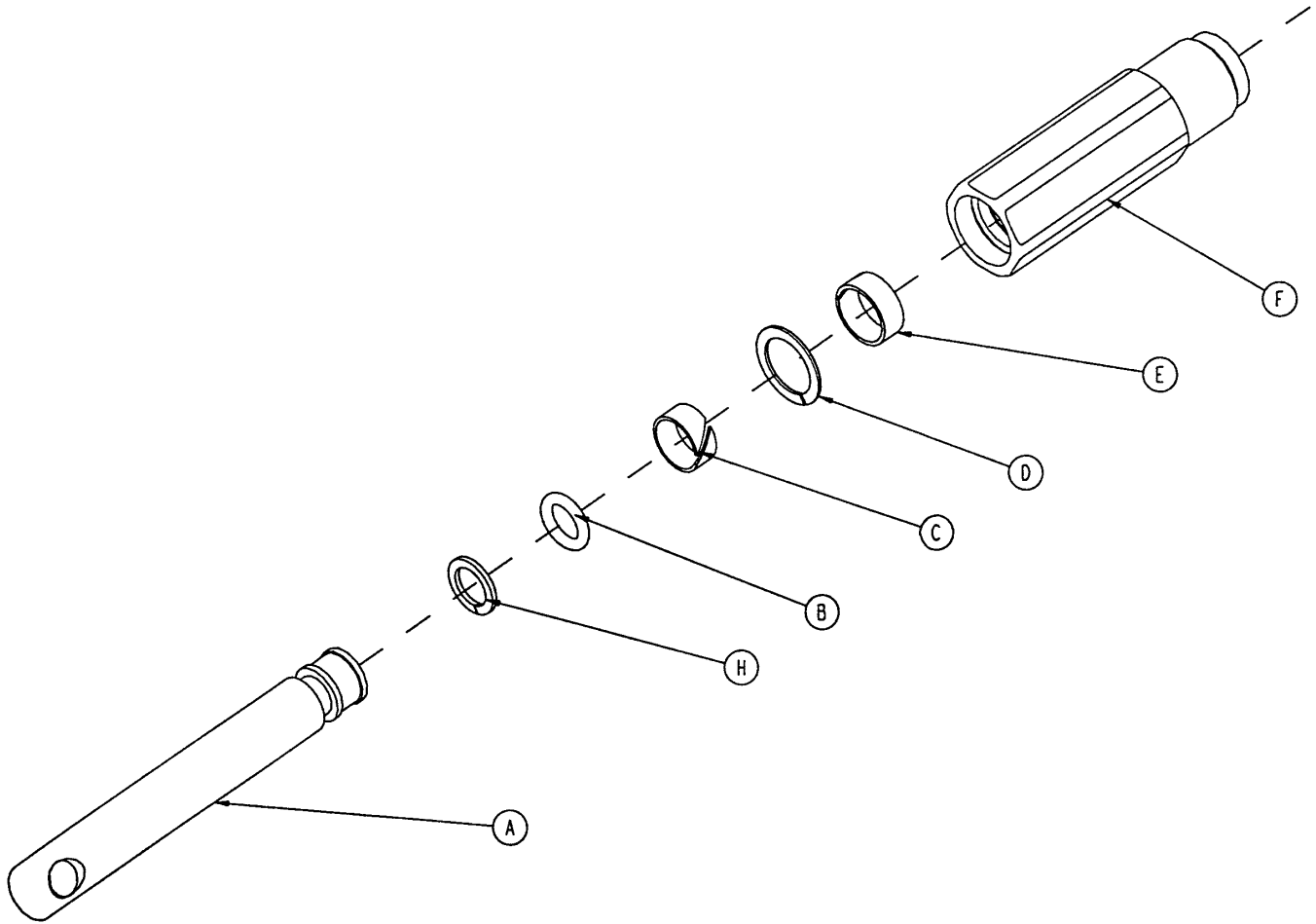

Item	Part No.	Part Name	Qty.
A	715–201–318	Pump Piston	1
B	45–110	O–Ring	1
C	715–1–328	Piston Wear Ring	1
D	14–50	Bearing Retainer	1
E	715–201–327	Cylinder Wear Ring	1
F	715–1–316	Pump Cylinder	1
H	715–1–400	O–Ring Back Up	1


Standard and Low Height Rack and Pinion Assembly

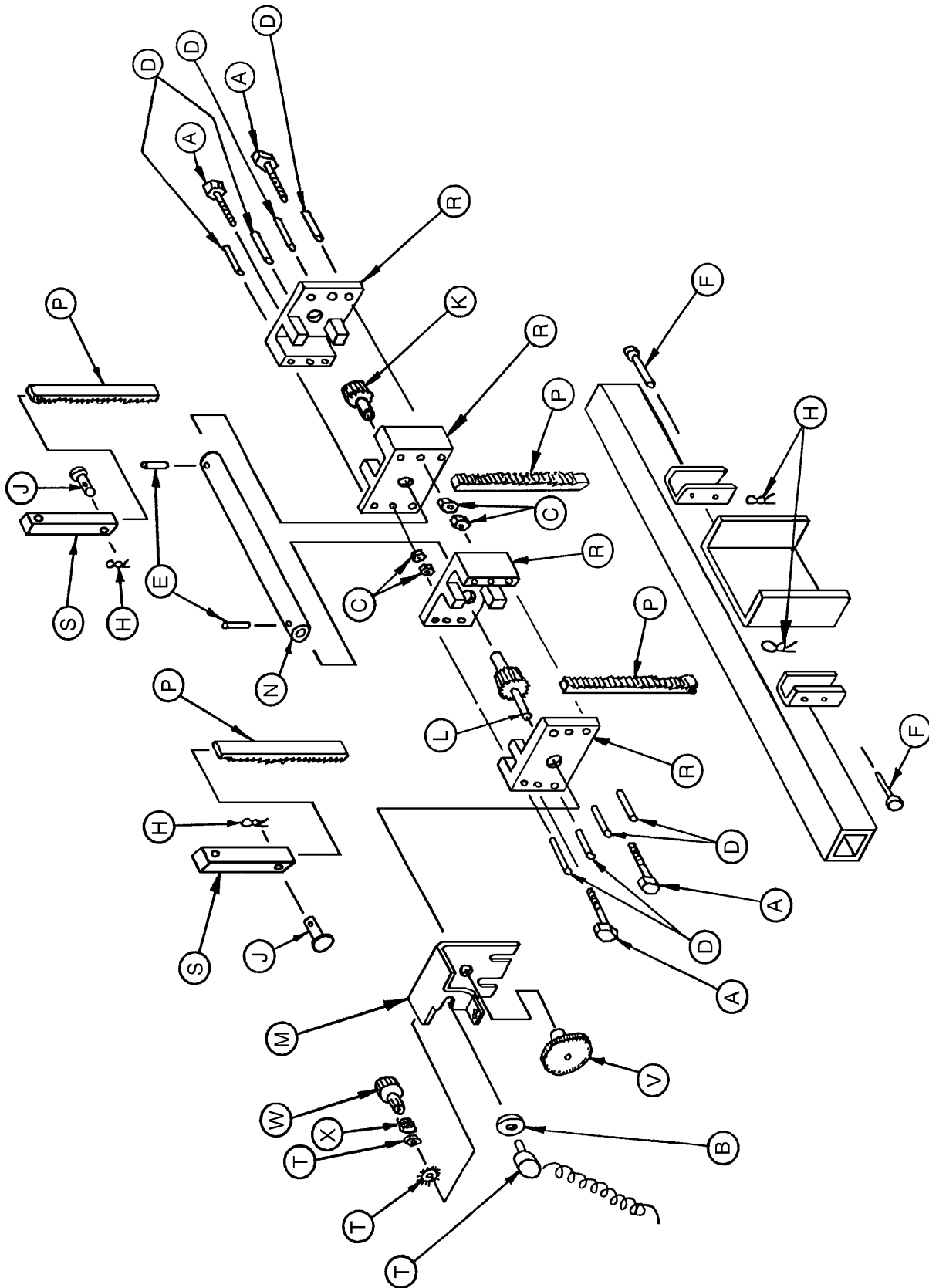


Standard and Low Height Rack and Pinion Assembly

2025-10-18				2025-10-17			
Low Height Rack & Pinion Assembly				Standard Height Rack & Pinion Assembly			
Item	Part No.	Part Name	Qty.	Item	Part No.	Part Name	Qty.
A	3-17	Hex Hd. Cap Screw	8	A	3-17	Hex Hd. Cap Screw	8
B	14-8	Washer	2	B	14-8	Washer	2
C	16-28	Nylock Nut	8	C	16-28	Nylock Nut	8
D	26-168	Spirol Pin	16	D	26-168	Spirol Pin	16
E	26-179	Roll Pin	4	E	26-179	Roll Pin	4
F	26-251	Pin	4	F	26-251	Clevis Pin	4
H	27-20	Rue Ring Cotter	8	H	27-20	Rue Ring Cotter	8
J	26-295	Clevis Pin	4	J	26-295	Clevis Pin	4
K	2025-1-61	Stabilizing Pinion	2	K	2025-1-61	Stabilizing Pinion	2
L	2025-1-64	Stabilizing Pinion & Pin	2	L	2025-1-64	Stabilizing Pinion & Pin	2
M	2025-1-67	Pot. Gear Bracket	2	M	2025-1-67	Pot. Gear Bracket	2
N	2025-1-68	Pinion Retaining Tube	2	N	2025-1-68	Pinion Retaining Tube	2
P	2025-1-70	Stabilizing Rack	8	P	2025-1-70	Stabilizing Rack	8
R	2025-1-71	Stabilizing Plate	8	R	2025-1-71	Stabilizing Plate	8
S	2025-10-23	Stabilizing Rack Sleeve	4	S	2025-1-100	Stabilizing Rack Sleeve	4
T	2025-1-807	Pot. Coil Cord	2	T	2025-1-807	Pot. Coil Cord	2
V	2025-201-66	Pot. Spur Gear	2	V	2025-201-66	Pot. Spur Gear	2
W	3000-200-215	Pot. Worm Gear	2	W	3000-200-215	Pot. Worm Gear	2
Y	3000-200-253	Pot. Worm Gear Retainer	4	X	3000-200-253	Pot. Worm Gear Retainer	4


Litter Assembly

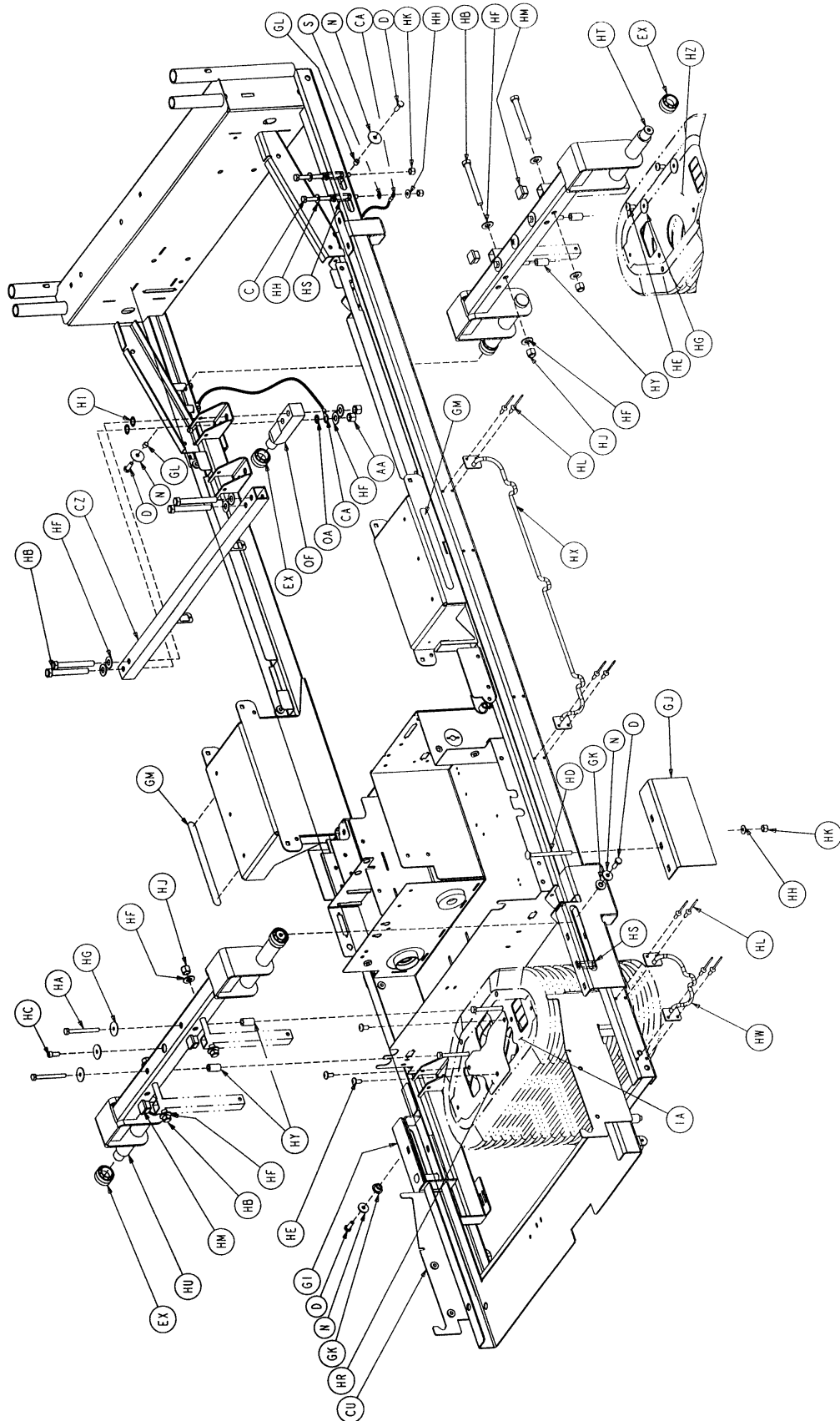


Litter Assembly

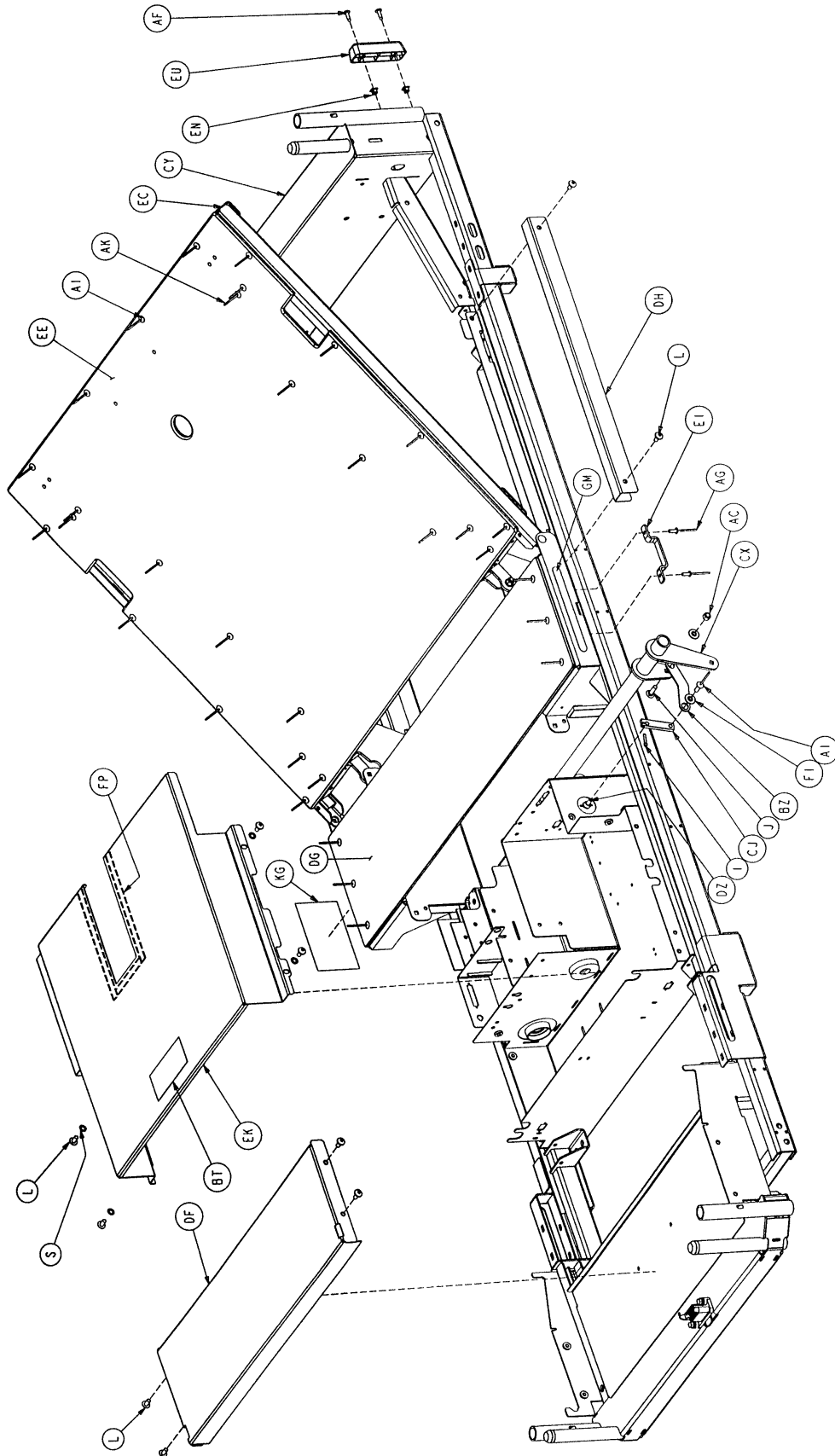


Litter Assembly

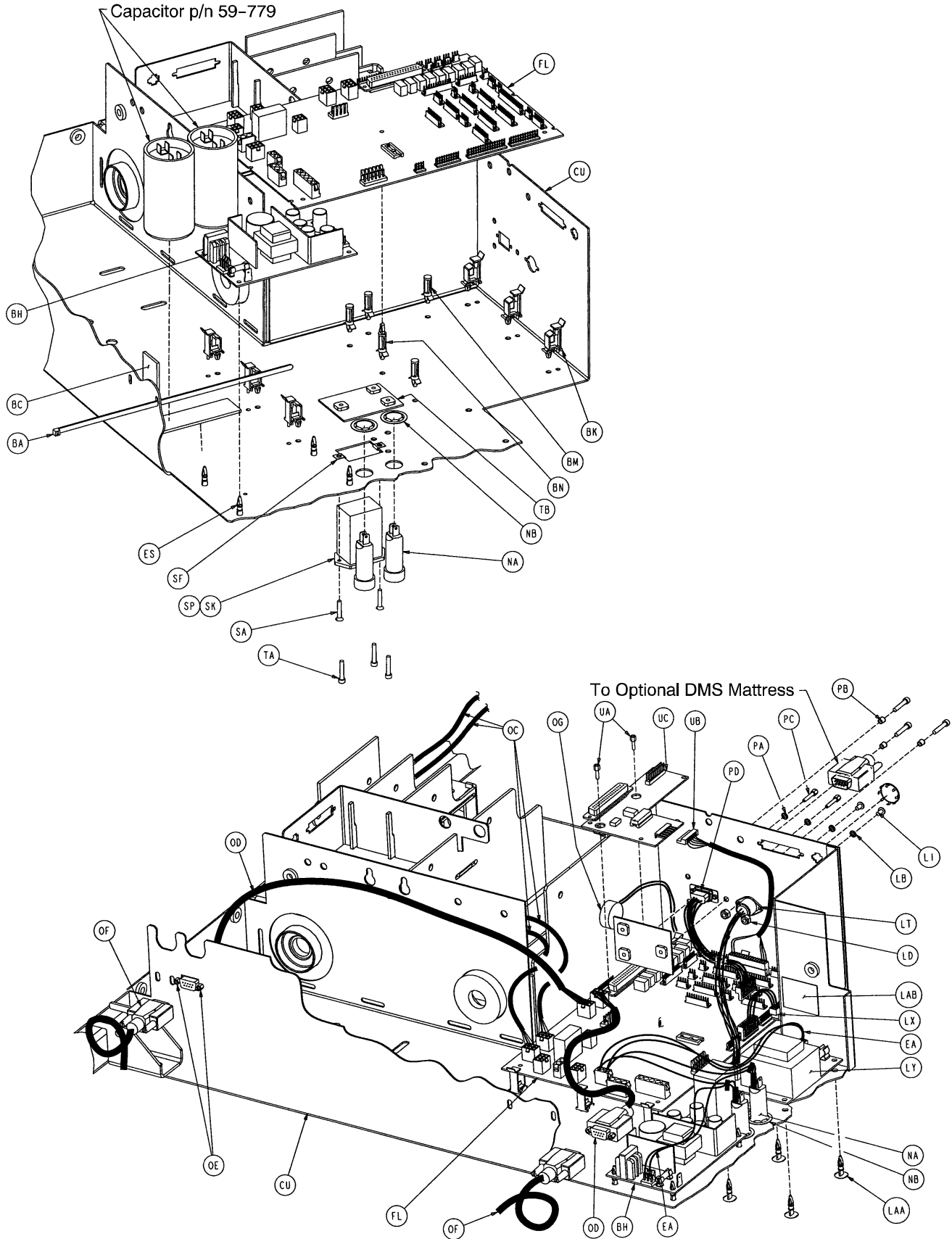


Litter Assembly

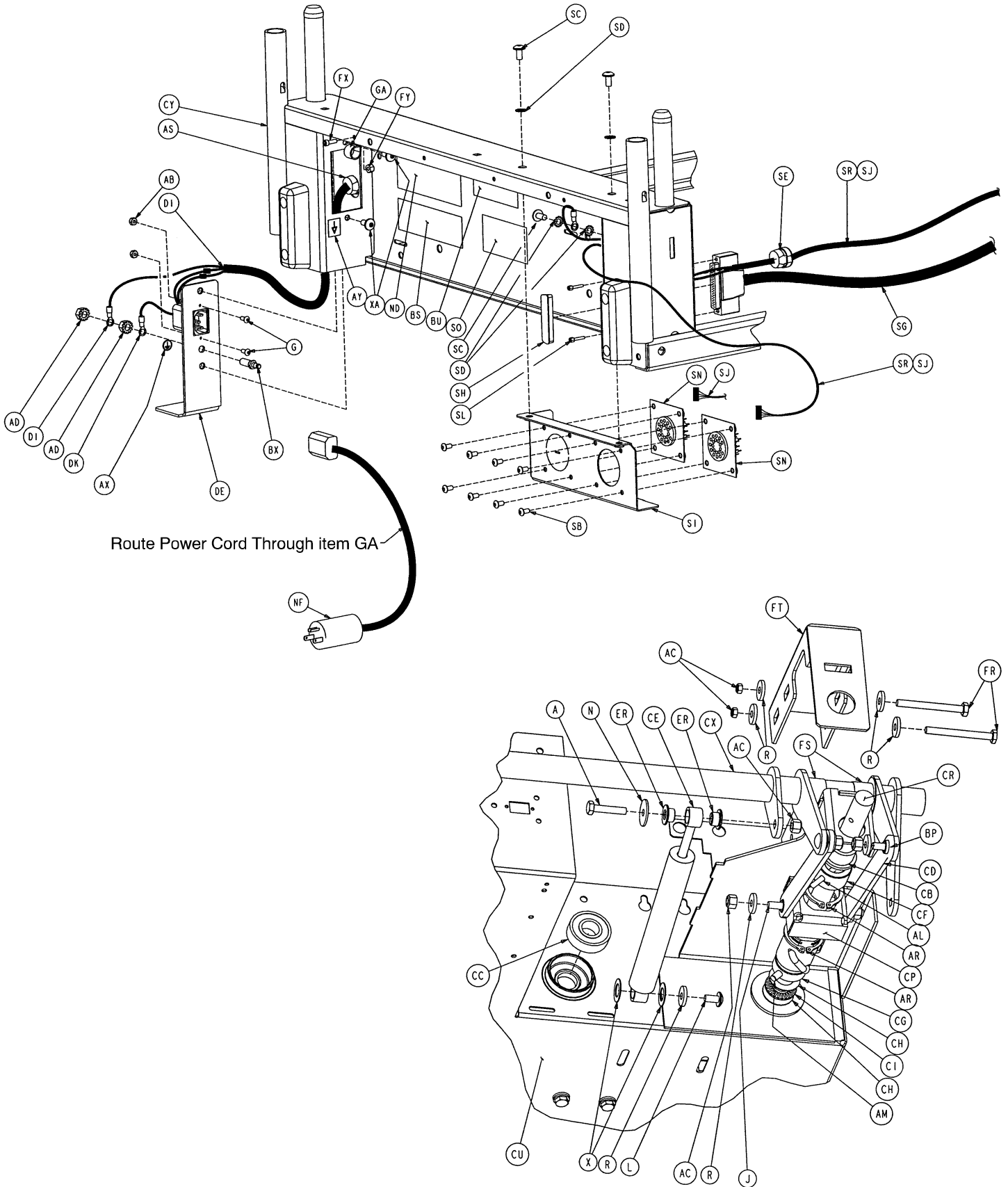


Litter Assembly

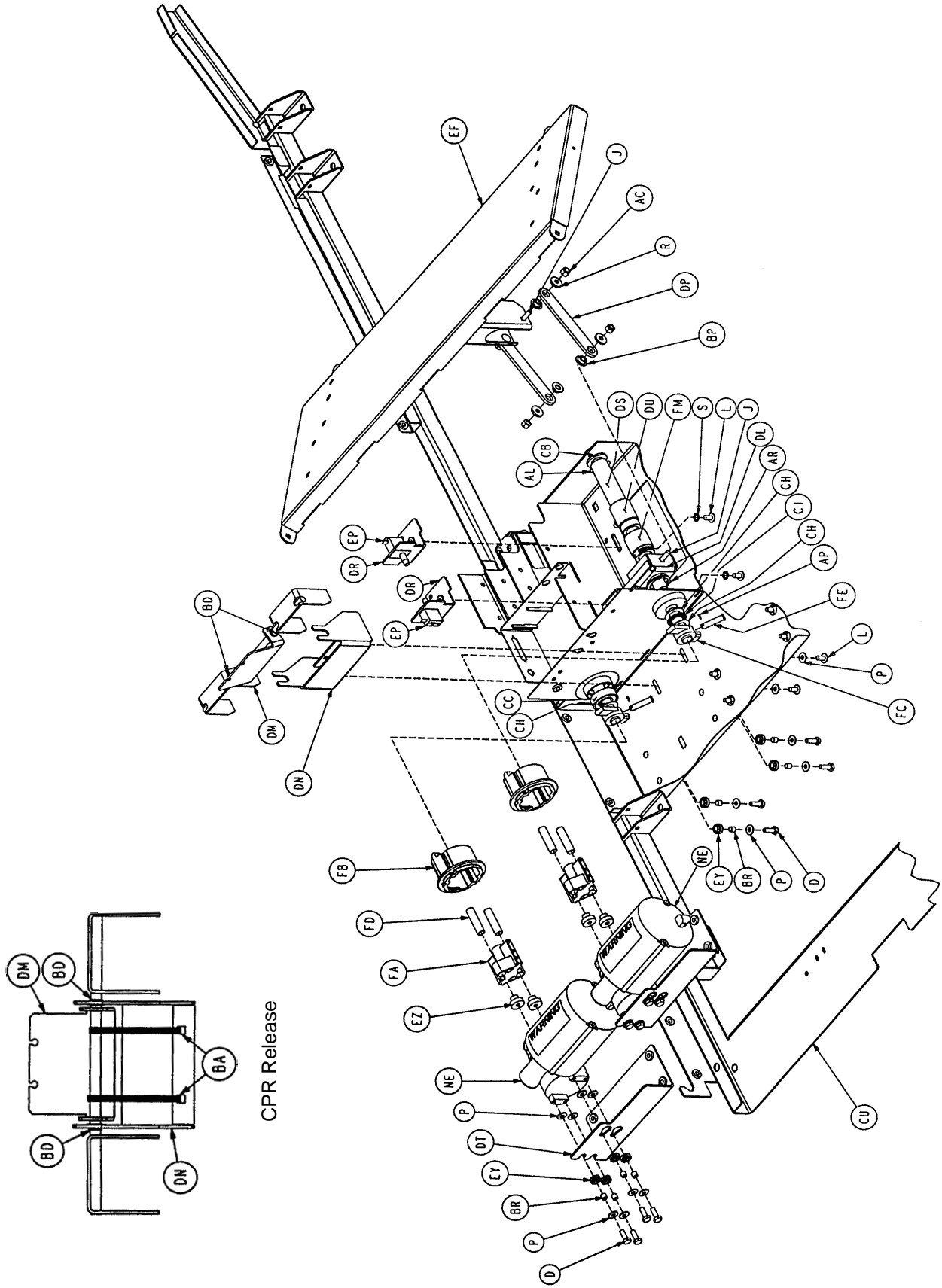



Integated StryKair Option Assembly
part number 2600-331-10

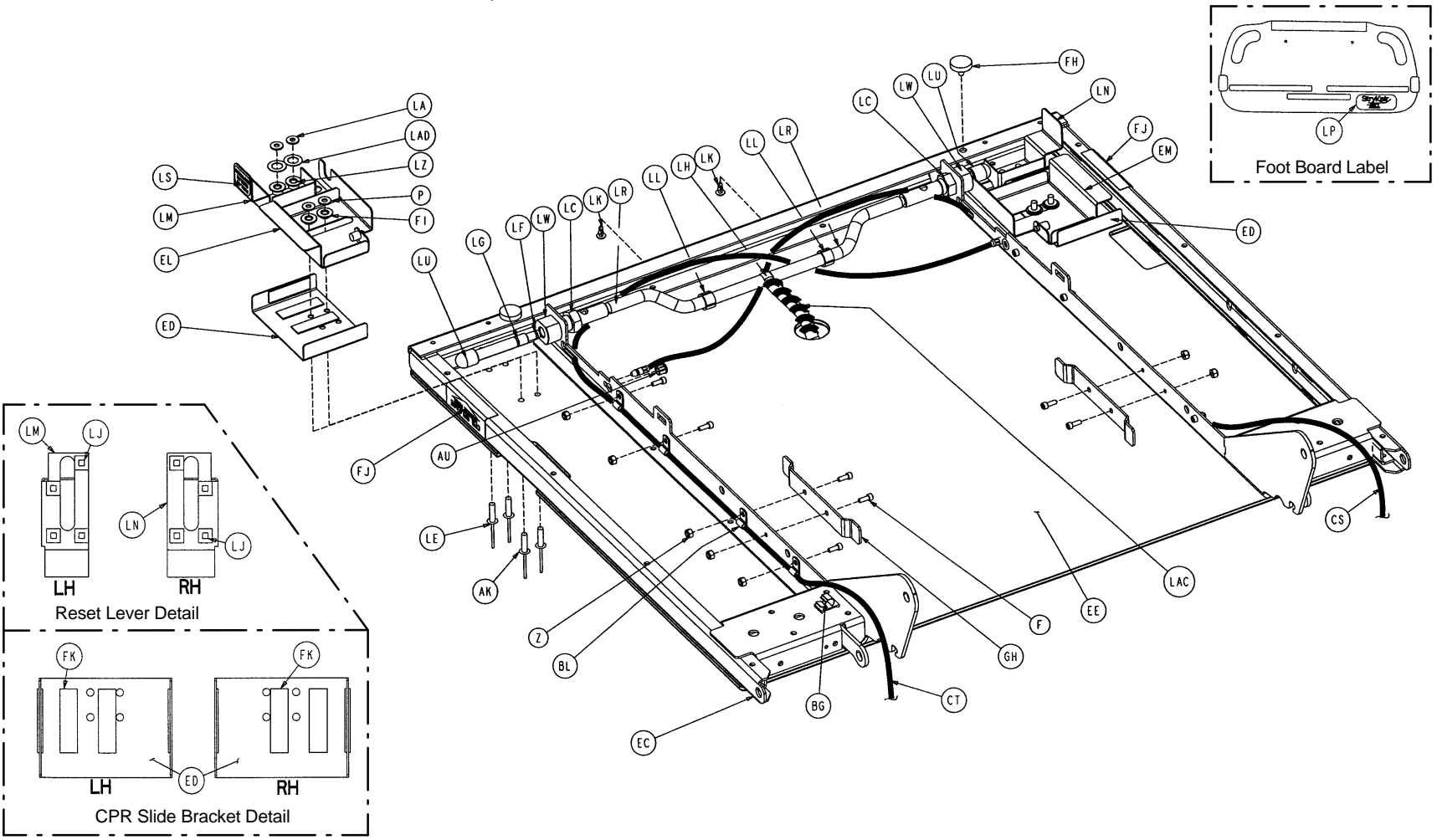

Litter Assembly

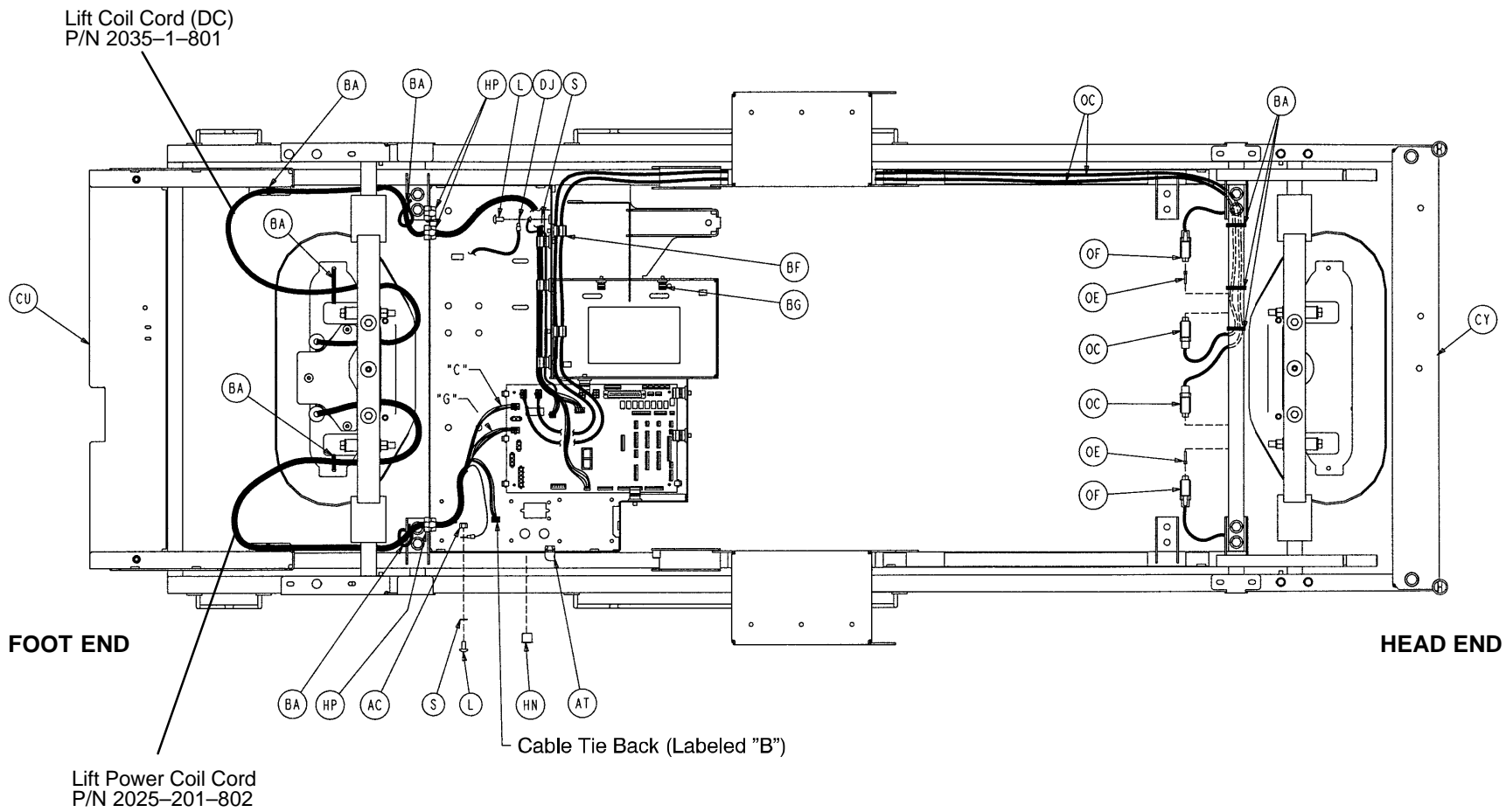

Litter Assembly

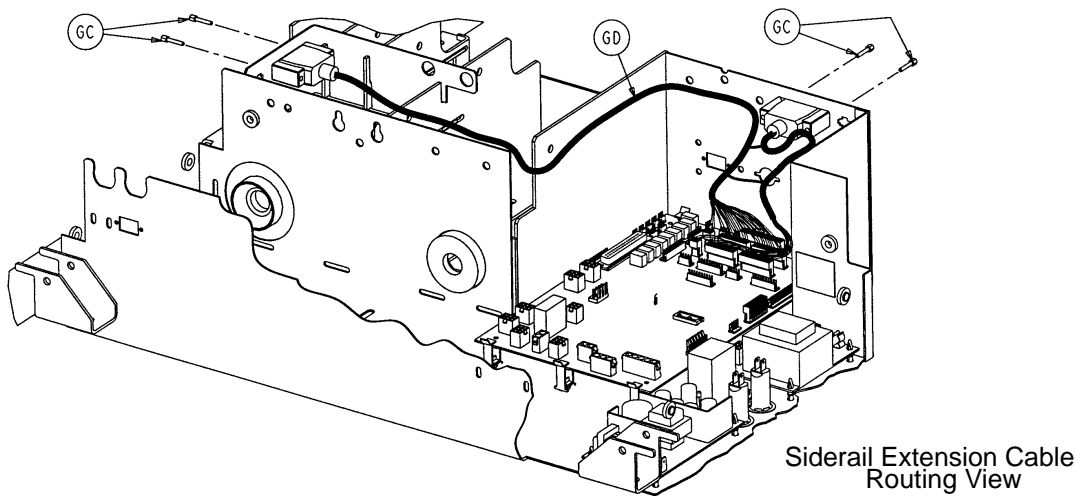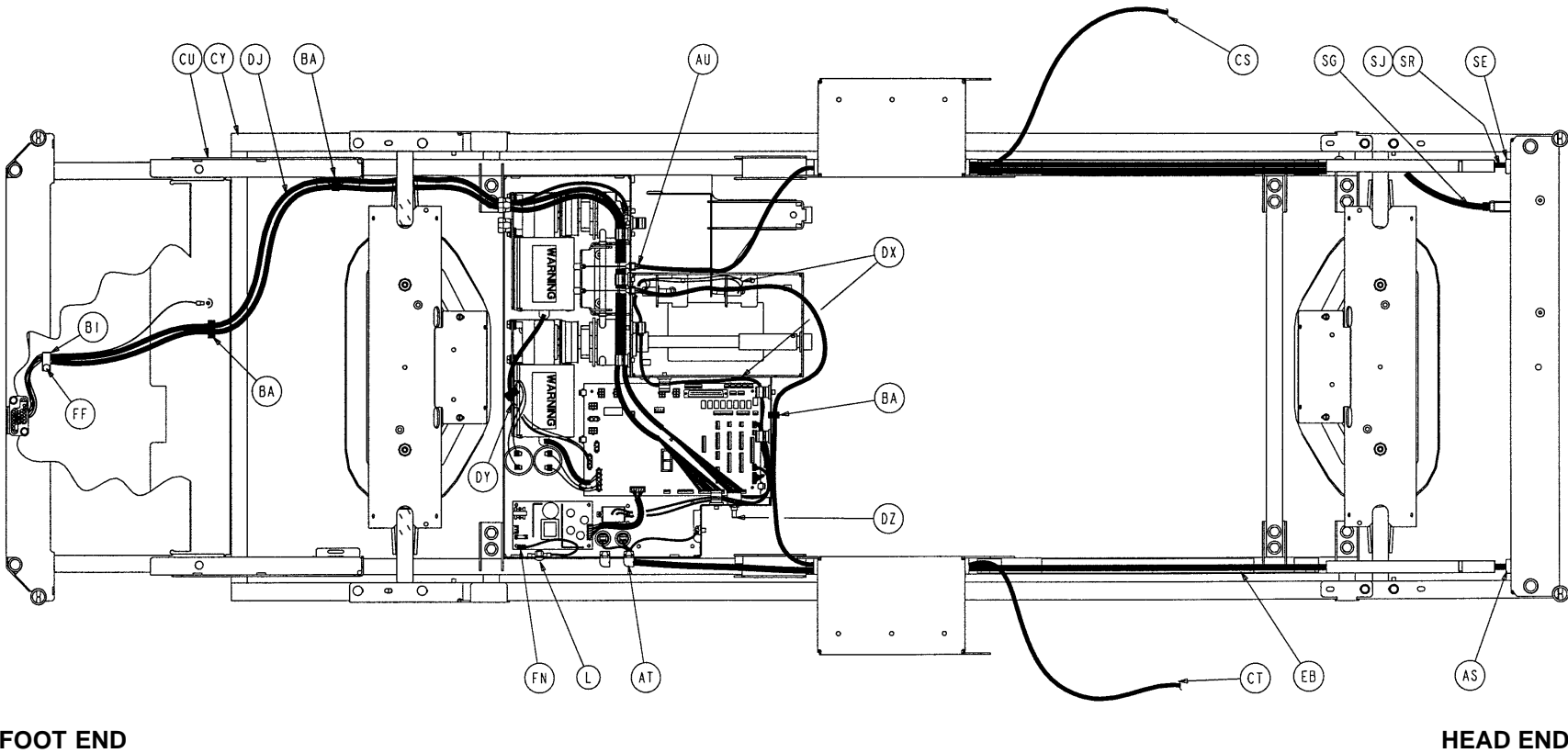

Litter Assembly

Item	Part No.	Part Name	Qty.	Item	Part No.	Part Name	Qty.
A	3-4	Hex Hd. Cap Screw	3	BU	2011-1-104	Danger Label	1
B	3-17	Hex Hd. Cap Screw	2	BX	2011-1-215	Grounding Lug	1
C	3-20	Hex Hd. Cap Screw	4	BY	2020-34-758	Calf Section Rest	2
D	3-214	Hex Hd. Cap Screw	20	BZ	2025-31-62	Pot. Actuator Link	1
E	3-347	Hex Hd. Cap Screw	4	CA	2025-31-804	Ground Jumper	4
F	4-32	Soc. Hd. Cap Screw	10	CB	2025-32-68	Flange Bearing	2
G	4-49	Soc. Hd. Cap Screw	2	CC	2025-32-76	Ball Bearing	2
H	4-85	Soc. Hd. Cap Screw	2	CD	2025-32-77	Fowler Actuator Link	2
I	4-101	Soc. Hd. Cap Screw	1	CE	2025-32-82	Hydraulic Dampener	2
J	5-19	Carriage Bolt	20	CF	2025-32-84	Fowler Screw Up Stop	1
K	5-23	Carriage Bolt	1	CG	2025-32-85	Fowler Screw Down Stop	1
L	7-58	Truss Hd. Torx	23	CH	2025-32-86	Thrust Washer	5
M	11-4	Washer	12	CI	2025-32-87	Big Roller Cage Bearing	2
N	11-53	Washer	8	CJ	2025-231-61	Pot. Timing Clamp	1
P	11-63	Washer	41	CK	2025-231-88	Fowler Link	2
R	11-158	Washer	18	CL	2025-231-90	Torque Tube Pivot Brg.	2
S	13-10	Ext. Tooth Lock Washer	12	CM	2025-231-112	Bed Extender Pin Lock	2
T	13-18	Ext. Tooth Lock Washer	14	CN	2025-31-805	Ground Strap	6
U	13-32	Ext. Tooth Lock Washer	2	CP	2025-232-89	Fowler Nut Box	1
X	14-7	Washer	2	CR	2025-232-90	Fowler Ball Screw	1
Y	14-8	Washer	4	CS	2035-31-48	Short CPR Cable Ass'y	1
Z	16-3	Nylock Nut	14	CT	2035-31-49	Long CPR Cable Ass'y	1
AA	16-35	Nylock Nut	8	CU	2035-31-50	Scale Frame Weldment	1
AB	16-23	Nylock Nut	2	CX	2035-31-51	Torque Tube Weldment	1
AC	16-28	Nylock Nut	25	CY	2035-31-54	Isolated Frame Wldmt.	1
AD	16-33	Kep Nut	2	CZ	2035-31-55	H/E Crosstube Wldmt.	1
AE	23-25	Hex Washer Hd. Screw	14	DA	2035-31-57	Bed Extender Weldment	1
AF	23-80	Truss Hd. Screw	4	DB	2035-31-64	Torq. Tube Ret. Brkt., Lt.	1
AG	25-50	Rivet	4	DC	2035-31-65	Torq. Tube Ret. Brkt., Rt.	1
AH	25-79	Rivet	2	DD	2035-31-66	Torque Block Channel	2
AI	25-142	Rivet	29	DE	2035-231-70	Mounting Plate A/C Filter	1
AK	25-147	Rivet	4	DF	2035-31-94	Foot Support Cover	1
AL	26-12	Roll Pin	2	DG	2035-31-97	Seat Section Skin	1
AM	26-168	Spiral Pin	2	DH	2035-31-100	Litter Wire Channel Cover	2
AN	27-15	Cotter Pin	4	DI	2035-31-801	Inlet Cable	1
AP	27-17	Cotter Pin	2	DJ	2035-31-802	Foot Board CPU Cable	1
AR	28-120	Ext. Retaining Ring	3	DK	2035-31-803	Power Inlet Cable	1
AS	30-27	Strain Relief	1	DL	2035-32-52	Gatch Trigger Weldment	1
AT	30-47	Right Angle Strain Relief	1	DM	2035-32-54	CPR Rel. Brkt. Wldmt.	1
AU	30-52	Snap Bushing	4	DN	2035-32-72	CPR Rel. Pivot Bracket	1
AX	36-46	220V Ground Label	1	DP	2035-32-77	Gatch Actuator Link	2
AY	36-115	Earth Ground Label	1	DR	2035-32-79	Act. Box Switch Bracket	2
AZ	38-111	Cable Tie	10	DS	2035-32-85	Gatch Screw Down Stop	1
BA	38-151	Long Cable Tie	17	DT	2035-32-88	Act. Box Motor Mtg. Brkt.	2
BB	38-382	Compression Spring	2	DU	2035-32-90	Gatch Ball Screw Ass'y	1
BC	44-29	Black Foam Tape	1	DX	2035-32-801	Gatch Limit Switch Cable	1
BD	52-759	Flange Bearing	2	DY	2035-32-802	Fow./CPU Jumper Cable	1
BE	58-56	Black Edge Trim	.5'	DZ	2035-32-803	Fowler Pot. Cable	1
BF	59-135	Big Push Mount Wire Clip	8	EA	2035-32-804	Fuse/PCB Cable	1
BG	8815-001-100	Wire Mount Clip	4	EB	2035-32-805	CPU/Power Supply Cable	1
BH	59-157	Power Supply	1	EC	2035-33-50	Fowler Frame Weldment	1
BI	59-743	Wire Harness Clip	2	ED	2035-33-62	CPR Rel. Slide Bracket	2
BK	59-751	Circuit Bd. Locking Supt.	6	EE	2035-33-63	Fowler Skin	1
BL	59-767	Cable Clamp	8	EF	2035-34-50	Thigh Section Weldment	1
BM	59-773	Push Spacer	4	EG	2035-35-50	Foot Section Weldment	1
BN	59-774	PCB Locking Support	1	EH	2035-35-96	Foot Prop Rod	1
BP	81-268	Flange Bearing	14	EI	2035-53-20	Restraint Strap Bracket	2
BR	715-1-333	Rel. Valve Stop Sleeve	16	EJ	2035-231-99	Bed Extender Rel. Lever	2
BS	1550-90-1	Hospital Grade Plug Label	1	EK	2035-232-75	Actuator Box Cover	1
BT	1550-90-13	Shock Caution Label	1	EL	2035-233-64	Quick Drop Rel. Brkt., Lt.	1

Litter Assembly

Item	Part No.	Part Name	Qty.	Item	Part No.	Part Name	Qty.
EM	2035-233-65	Quick Drop Rel. Brkt., Rt.	1	FM	5000-30-366	Fowler Nut Adapter	1
EN	3000-300-2	Plastic Clip Nut	4	FN	5010-80-7	Power Supply Gd. Cable	1
EP	3000-300-58	Switch Plunger	2	FP	8800-380-000	Neoprene Sponge	1.5'
ER	3000-300-99	Fowler Modified Bushing	10	FR	3-23	Hex Hd. Cap Screw	2
ES	3000-300-115	Standoff	4	FS	52-762	Nyliner Bushing	2
ET	3000-300-349	Hd/Ft Board Post Cap		FT	2035-32-96	Ball Screw Cover	14
EU	3000-300-350	Head End Bumper Strip	2	FX	4-5	Soc. Hd. Cap Screw	1
EX	3000-300-353	Roller	8	FY	16-14	Nylock Nut	1
EY	3000-300-442	Fowler Drive Grommet	16	FZ	16-102	Nylock Jam Nut	3
EZ	3000-300-455	CPR Isolation Bushing	4	GA	34-22	Cord Clamp	1
FA	3000-300-456	CPR Isolator	2	GC	59-727	Jack Screw	4
FB	3000-300-461	CPR Decoupler	2	GD	3001-300-877	Siderail Extension Cable	1
FC	3000-300-462	CPR Wing	2	GE	59-133	Push-Mount Wire Clip	1
FD	3000-300-464	CPR Engagement Spring	4	GF	44-32	1" Poly Tape	50"
FE	3000-300-473	Clevis Pin	2	GG	2035-32-84	Gatch Screw Up Stop	1
FF	58-76	Drive Fastener	2	GH	2035-400-565	Siderail Guide Bracket	2
FG	3001-200-228	Mounting Standoff	2	GI	2035-31-115	Roller Bracket Cover, Rt.	1
FH	3001-300-8	Thigh Section Bumper	4	GJ	2035-31-116	Roller Bracket Cover, Lt.	1
FI	3001-300-99	Flange Bearing	10	GL	715-1-333	Stop Sleeve	2
FJ	3001-300-603	CPR Release Label	2	GM	6080-90-16	"500 Lb. Max" Label	2
FK	3001-300-663	Velcro Strip	10	XA	7-58	Truss Hd. Torx	2
FL	*	CPU Board	1				

* 2035-700-1 – CPU Kit – Scale Only

2035-700-2 – CPU Kit – Bed Exit Only

2035-700-3 – CPU Kit – Scale and Bed Exit

2035-32-10 Apex Litter Components

Item	Part No.	Part Name	Qty.	Item	Part No.	Part Name	Qty.
HA	3-23	Hex Hd. Cap Screw	4	HM	37-22	Square Plug	4
HB	3-83	Hex Hd. Cap Screw	8	HN	37-202	Round Plug	1
HC	4-14	Soc. Hd. Cap Screw	2	HP	59-106	Strain Relief	3
HD	5-29	Rd. Hd. Sq. Neck Bolt	2	HR	2025-31-117	Bellows Ret. Plate, Foot	1
HE	7-58	Truss Hd. Torx	5	HS	3001-300-4	Spacer	6
HF	11-4	Washer	16	HT	2035-231-52	Head End Header Wldmt.	1
HG	11-53	Washer	8	HU	2035-231-53	Foot End Header Wldmt.	1
HH	11-63	Washer	8	HW	2035-31-56	Short Foley Weldment	2
HI	13-32	Ext. Tooth Lock Washer	4	HX	2035-31-58	Long Foley Weldment	2
HJ	16-35	Nylock Nut	8	HY	3000-300-428	Gatch Link Sleeve	4
HK	16-28	Nylock Nut	6	HZ	2025-31-115	Bellows Bracket, Hd. End	1
HL	25-50	Rivet	16	IA	2025-31-116	Bellows Bracket, Ft. End	1

2035-32-15 Litter Domestic Components

Item	Part No.	Part Name	Qty.
NA	59-179	Circuit Breaker	2
NB	59-181	Clip Retainer	2
NC	59-779	Capacitor	2
ND	2035-31-125	Specification Label	1
NE	2035-300-705	Fowler/Gatch Motor Ass'y	2
NF	39-254	Power Cord	1

2036-32-15 Litter International Components

Item	Part No.	Part Name	Qty.
NA	59-178	Circuit Breaker	2
NB	59-181	Clip Retainer	2
NC	59-153	Capacitor	2
ND	2036-31-125	Specification Label	1
NE	3221-300-705	Fowler/Gatch Motor Ass'y	2

2036-32-16 Litter European Comp. – No Scale

Item	Part No.	Part Name	Qty.
NA	59-178	Circuit Breaker	2
NB	59-181	Clip Retainer	2
NC	59-153	Capacitor	2
ND	2036-231-126	Specification Label	1
NE	3221-300-705	Fowler Drive Assembly	2

2036-32-17 Litter European Comp. – Scale

Item	Part No.	Part Name	Qty.
NA	59-178	Circuit Breaker	2
NB	59-181	Clip Retainer	2
NC	59-153	Capacitor	2
ND	2036-231-127	Specification Label	1
NE	3221-300-705	Fowler Drive Assembly	2

Litter Assembly

2600–331–10 Litter Assembly – Integrated StryKair Option

Item	Part No.	Part Name	Qty.	Item	Part No.	Part Name	Qty.
LA	11–63	Washer	4	LP	2600–22–46	Foot Board Logo Label	1
LB	12–30	Ext. Tooth Star Washer	2	LR	2600–40–1	Hose	2
LC	15–61	Hex Jam Nut	2	LS	2600–90–99	Reset Label	2
LD	16–23	Nylock Nut	2	LT	2600–242–801	StryKair Port Cable	1
LE	25–147	Pop Rivet	4	LU	2600–300–98	Plunger	2
LF	45–993	O–Ring	2	LW	2600–300–99	Guide Tube	2
LG	45–994	O–Ring	2	LX	2600–300–802	StryKair Signal Cable	1
LH	48–171	Tee Fitting	1	LY	2600–300–900	StryKair Power Supply	1
LI	50–38	Pan Hd. Mach. Screw	2	LZ	3000–300–99	Bushing	4
LJ	58–75	Flat Tie Holder	8	LAA	3000–300–115	Standoff	4
LK	58–76	Drive Fastener	2	LAB	2600–320–606	Fuse Label	1
LL	59–743	Wire Harness Clip	2	LAC	34–273	Slit Harness Wrap	24"
LM	2035–233–66	Left Reset Lever	1	LAD	14–7	Washer	4
LN	2035–233–67	Right Reset Lever	1				

2035–30–100 No Scale or Bed Exit Options

Item	Part No.	Part Name	Qty.
	(page 11–76)	Foot Board, No Scale/BE	1
OF	3001–300–511	"Imitation" Load Cell	4

2035–30–125 Scale Option Only

Item	Part No.	Part Name	Qty.
OA	13–32	Ext. Tooth Lock Washer	4
	(page 11–78)	Foot Board, Scale Option	1
OC	2035–31–805	Load Cell Cable, Head	2
OD	2035–31–804	Load Cell Cable, Foot	2
OE	3001–300–7	M/F Screw	8
OF	3001–307–57	Load Cell	4

2035–30–175 Bed Exit Option Only

Item	Part No.	Part Name	Qty.
OA	13–32	Ext. Tooth Lock Washer	4
	(page 11–77)	Foot Board, BE Option	1
OC	2035–31–805	Load Cell Cable, Head	2
OD	2035–31–804	Load Cell Cable, Foot	2
OE	3001–300–7	M/F Screw	8
OF	3001–307–57	Load Cell	4
OG	3001–508–869	BE Beeper	1

2035–30–150 Scale w/Bed Exit Options

Item	Part No.	Part Name	Qty.
OA	13–32	Ext. Tooth Lock Washer	4
	(page 11–79)	Foot Board, Scale & BE	1
OC	2035–31–805	Load Cell Cable, Head	2
OD	2035–31–804	Load Cell Cable, Foot	2
OE	3001–300–7	M/F Screw	8
OF	3001–307–57	Load Cell	4
OG	3001–508–869	BE Beeper	1

2035–42 Integrated DMS Option

Item	Part No.	Part Name	Qty.
PA	12–30	Ext Tooth Lock Washer	2
PB	37–74	Hole Plug	3
PC	59–727	Jack Screw	2
PD	2035–42–801	DMS Port Cable	1

2035–30–205 No Nurse Call Option

Item	Part No.	Part Name	Qty.
TA	4–101	Soc. Hd. Cap Screw	3
TB	2025–32–74	DMS Cover Plate	1

2035–30–206 Smart TV Option

Item	Part No.	Part Name	Qty.
UA	59–176	Jack Screw	2
UB	2035–232–806	STV Option Control Cable	1
UC	3001–330–950	STV Comm. Board	1

2035–30–207 Smart TV W/Relays Option

Item	Part No.	Part Name	Qty.
UA	59–176	Jack Screw	2
UB	2035–232–806	STV Option Control Cable	1
UC	3001–330–970	STV Comm. Board	1

Litter Assembly

2035-30-200 Head Wall Communication Option

Item	Part No.	Part Name	Qty.
SG	59-175	Head Wall Interface Cable	1
SH	59-710	Static Cap	1
SL	3001-300-7	M/F Screw	2

2035-30-201 Head Wall Comm. w/Nurse Call

Item	Part No.	Part Name	Qty.
SA	1-87	Flat Hd. Mach. Screw	2
SF	52-783	U Clip	2
SG	59-175	Head Wall Interface Cable	1
SH	59-710	Static Cap	1
SK	3000-303-871	Battery	1
SL	3001-300-7	M/F Screw	2
SP	5010-80-20	9V Battery Box w/Cable	1


2035-30-202 HW Comm. w/NC & 1 Stryker Port

Item	Part No.	Part Name	Qty.
SA	1-87	Flat Hd. Mach. Screw	2
SB	4-307	But. Hd. Cap Screw	4
SC	7-58	Truss Hd. Torx	3
SD	13-10	Ext. Tooth Lock Washer	4
SE	3-27	Strain Relief Grommet	1
SF	52-783	U Clip	2
SG	59-175	Head Wall Interface Cable	1
SH	59-710	Static Cap	1
SI	2035-30-99	Pend. Port Head Wall Brkt.	1
SJ	2035-30-804	Pendant Port Cable	1
SK	3000-303-871	Battery	1
SL	3001-300-7	M/F Screw	2
SN	3001-314-920	Head Wall Pend. Port PCB	1
SO	5000-90-28	Cord Out Label	1
SP	5010-80-20	9V Battery Box w/Cable	1

2035-30-203 HW Comm. w/NC & 2 Stryker Ports

Item	Part No.	Part Name	Qty.
SA	1-87	Flat Hd. Mach. Screw	2
SB	4-307	But. Hd. Cap Screw	8
SC	7-58	Truss Hd. Torx	3
SD	13-10	Ext. Tooth Lock Washer	4
SE	3-27	Strain Relief Grommet	1
SF	52-783	U Clip	2
SG	59-175	Head Wall Interface Cable	1
SH	59-710	Static Cap	1
SI	2035-30-99	Pend. Port Head Wall Brkt.	1
SJ	2035-30-805	Pend. Port Cable, 2 Ports	1
SK	3000-303-871	Battery	1
SL	3001-300-7	M/F Screw	2
SN	3001-314-920	Head Wall Pend. Port PCB	2
SO	5000-90-28	Cord Out Label	1
SP	5010-80-20	9V Battery Box w/Cable	1

3001–300–775 Fowler Brake Kit Assembly

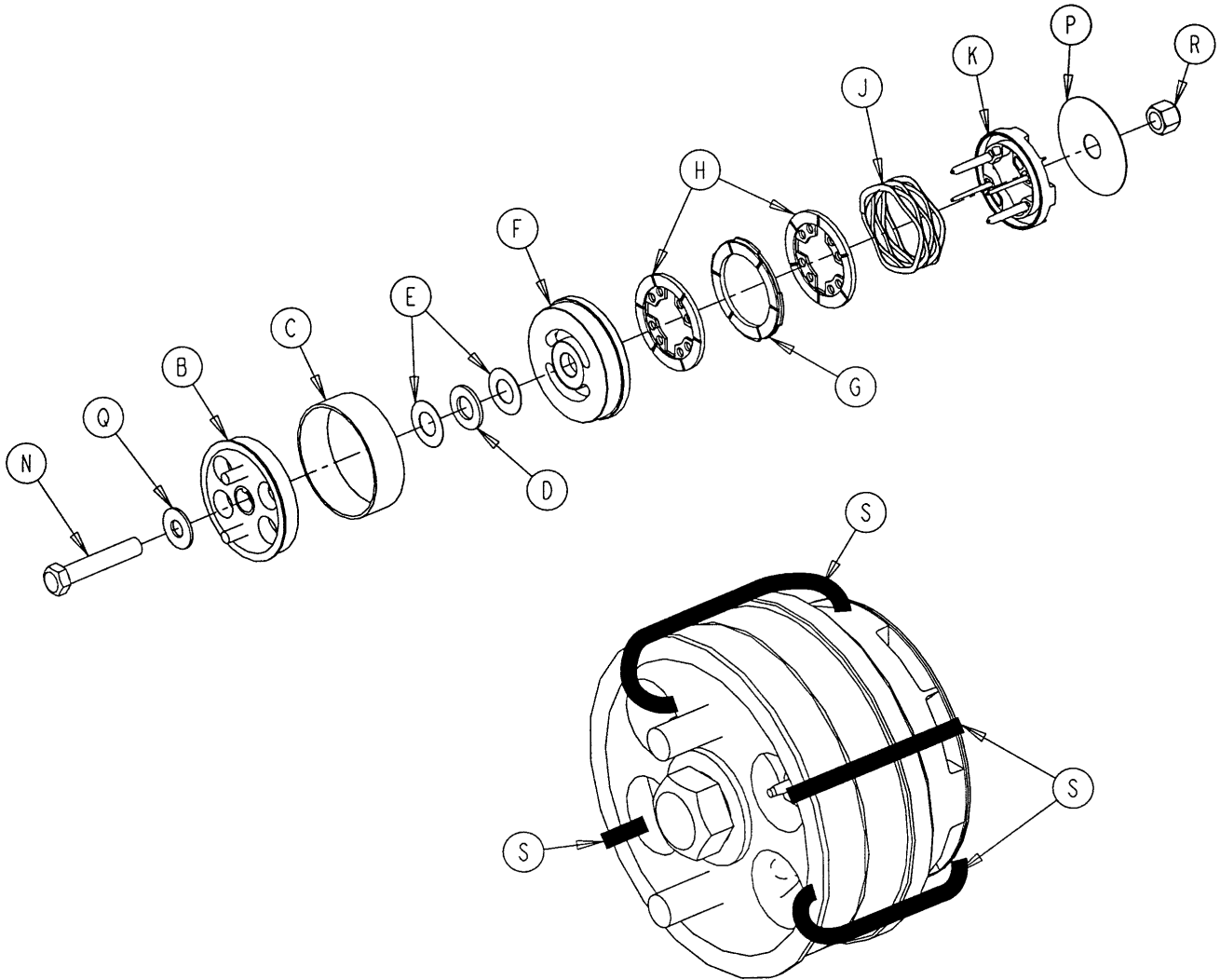


Item	Part No.	Part Name	Qty.
B	3001–300–455	CPR Coupler Assembly	1
C	3000–300–465	CPR Clutch Spring	1
D	3000–200–225	CPR Thrust Bearing	1
E	3000–200–224	Idler Gear Thrust Washer	2
F	3001–300–569	Brake Cup	1
G	3001–300–552	CPR Brake Disc	1
H	3001–300–551	CPR Spring Cup	2
J	3001–300–563	CPR Brake Spring	1
K	3001–300–570	CPR Spring Cup	1
N	3–64	Hex Hd. Cap Screw	1
P	3000–300–776	Flat Washer	1
Q	11–193	Heavy Flat Washer	1
R	16–35	Nylock Nut	1
S	3000–300–113	8" Wire Tie	4


Notes

Head End Siderail Assembly


Assembly part number
2035-400-105 (Left) & 2035-400-205 (Right)
(reference only)

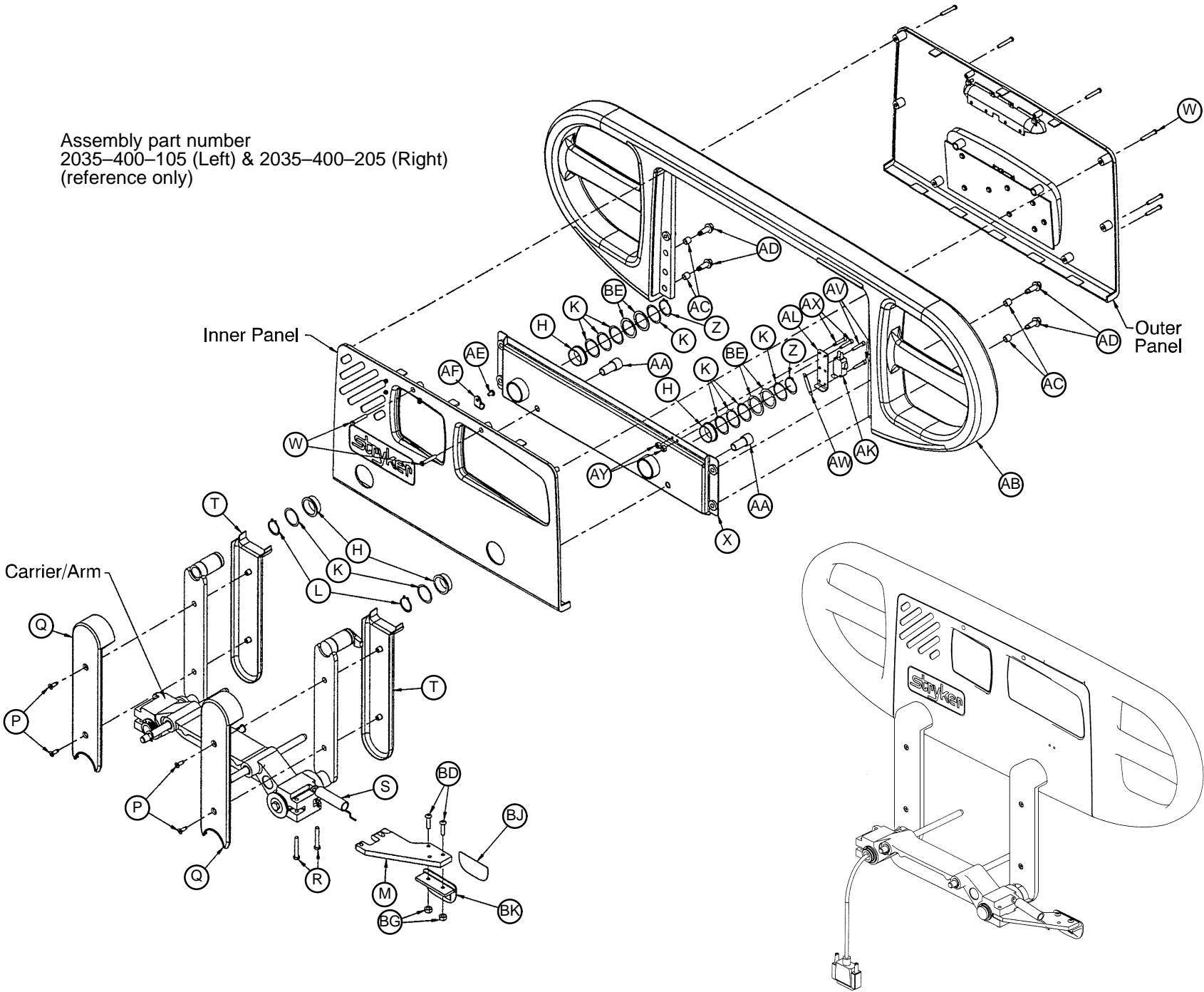

Head End Siderail Assembly

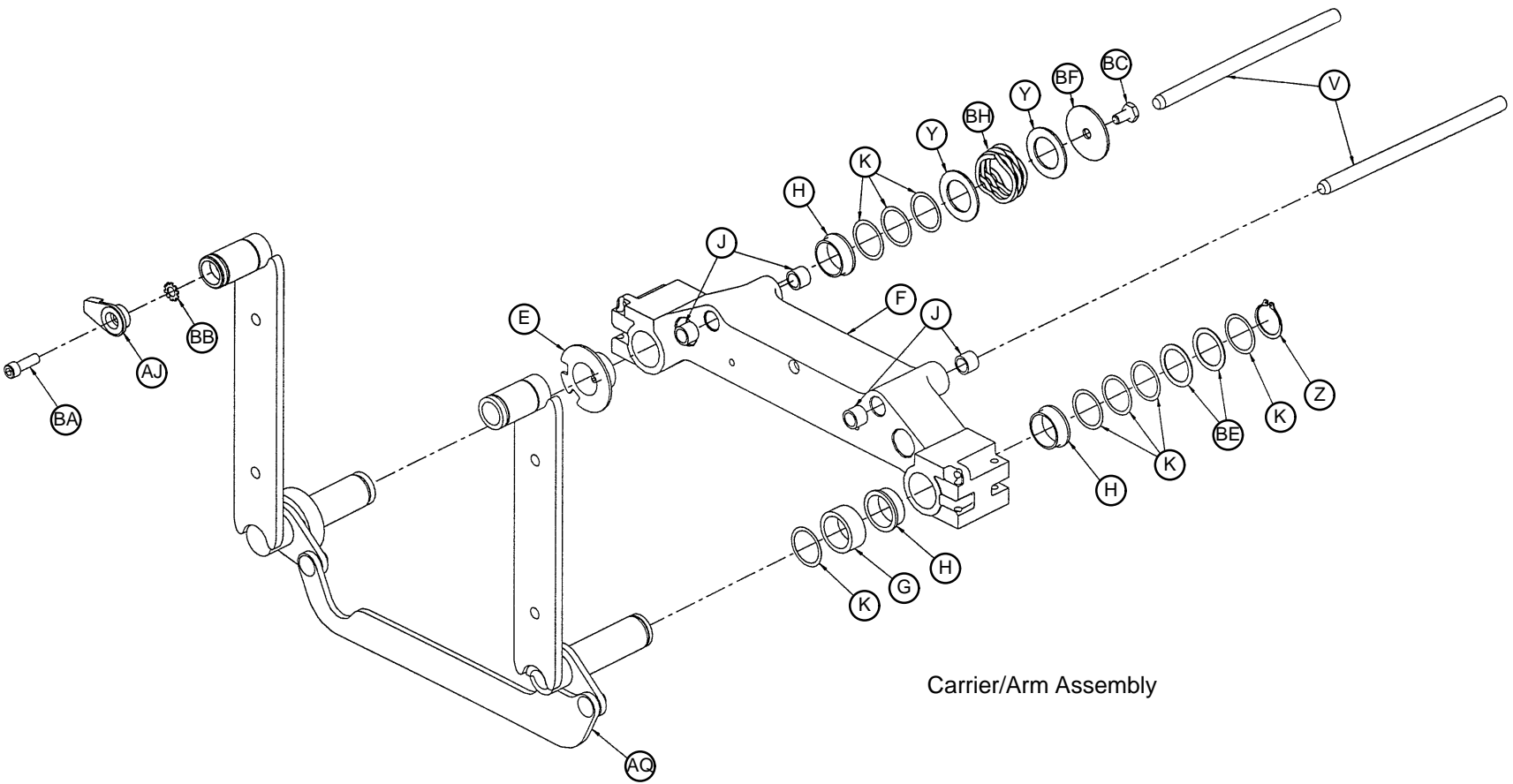


Head End Siderail Assembly

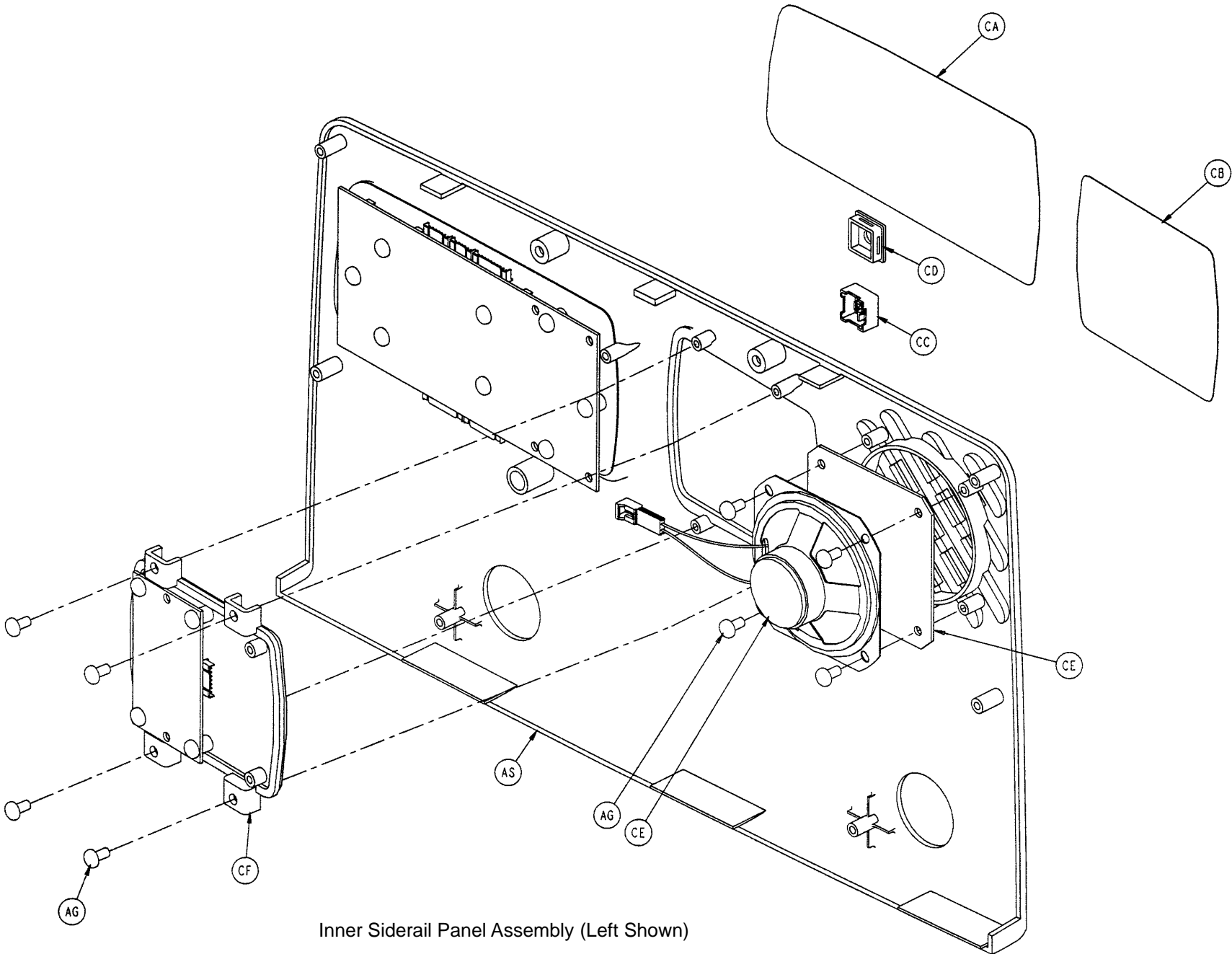


Inner Siderail Panel Assembly (Left Shown)

Head End Siderail Assembly

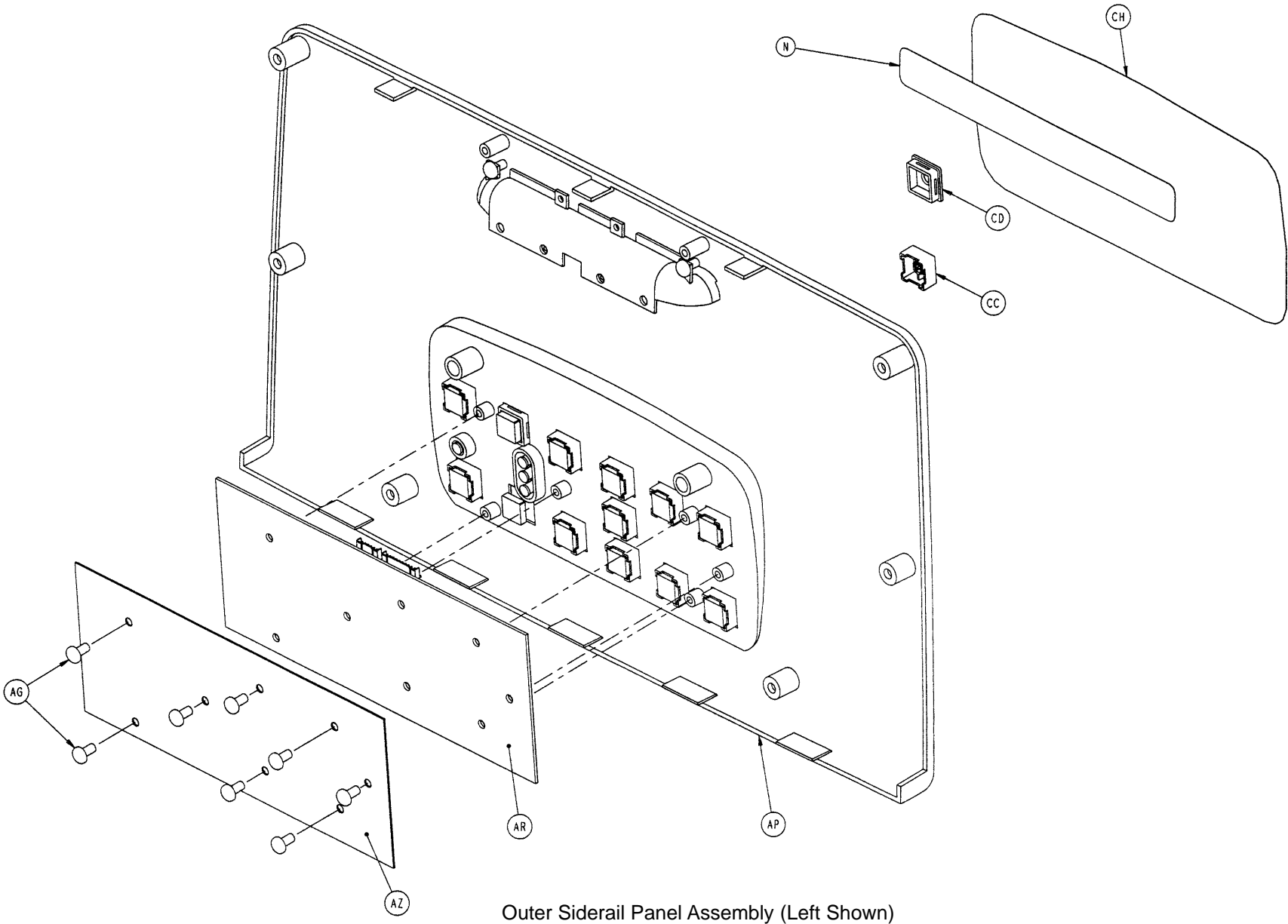


Head End Siderail Assembly

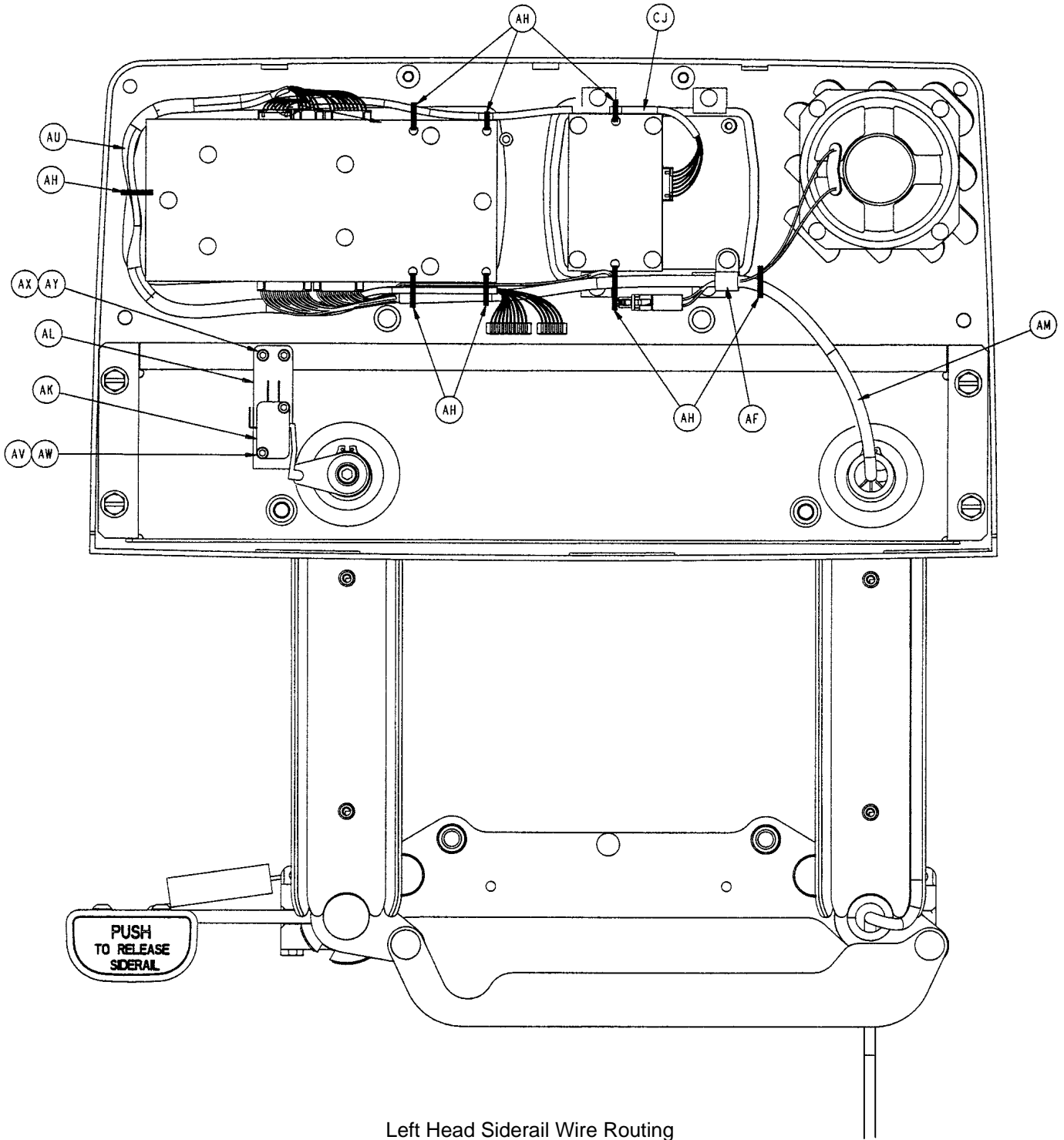


Left Head Siderail Wire Routing

Head End Siderail Assembly

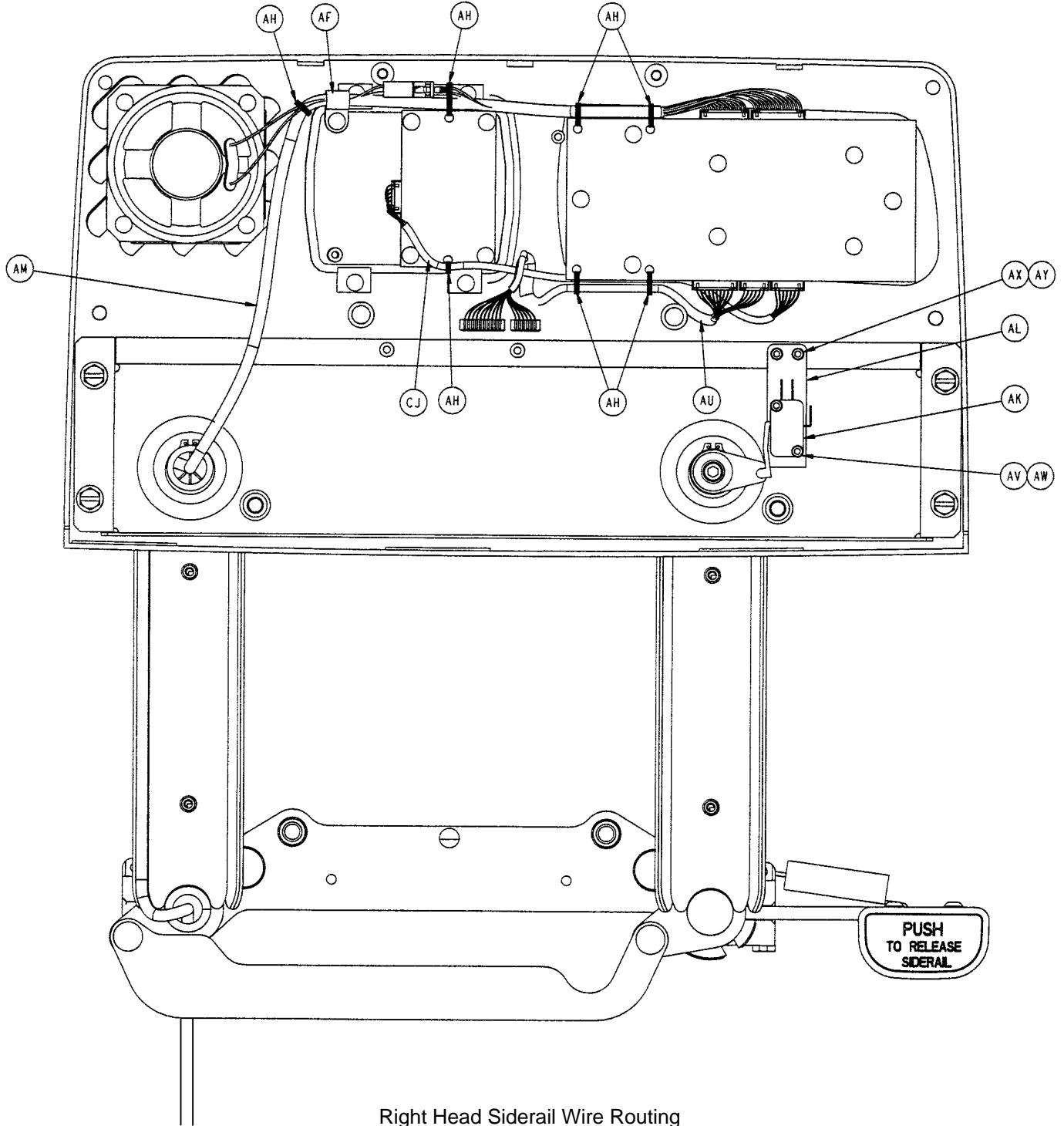


Right Head Siderail Wire Routing

Head End Siderail Assembly

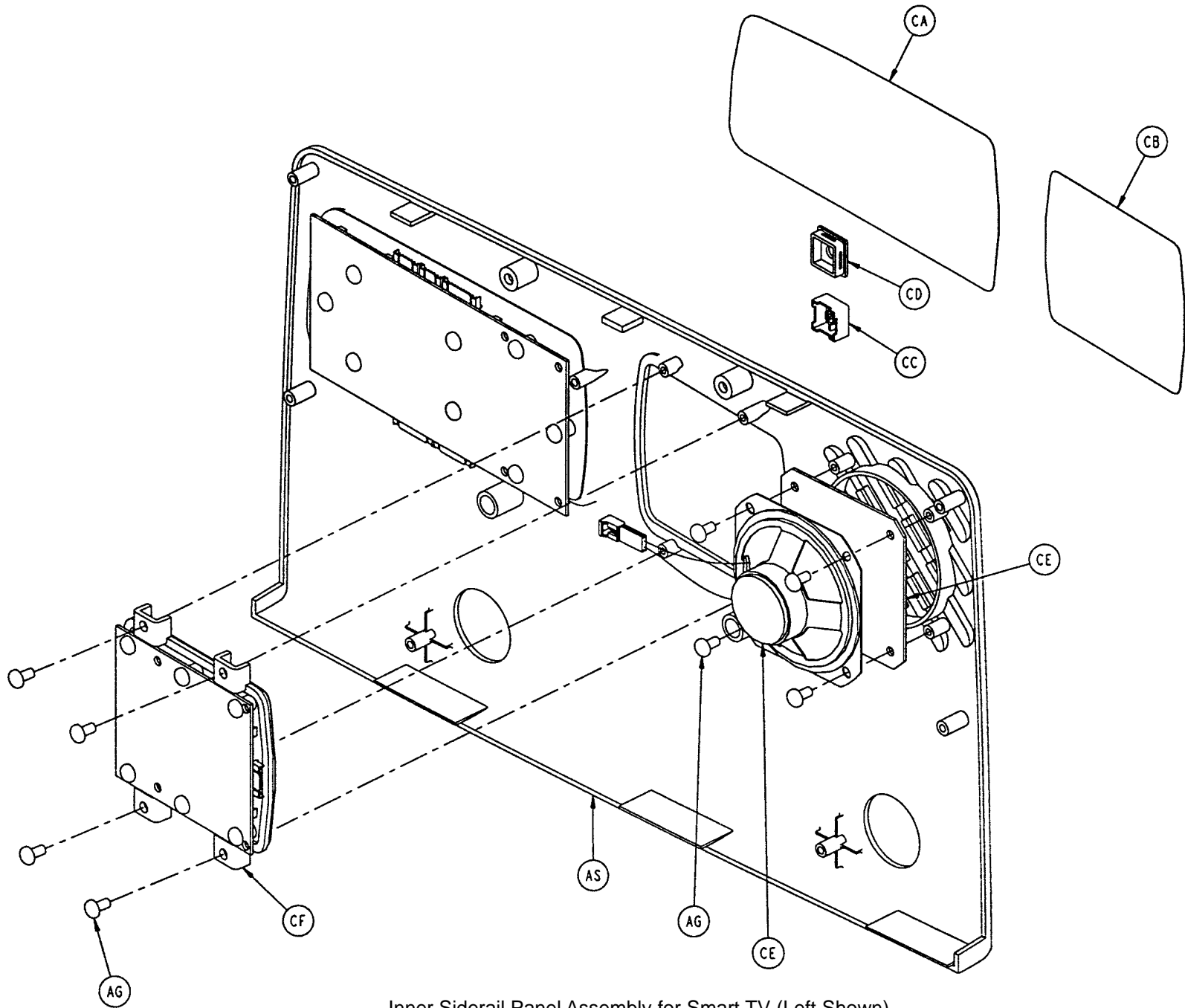


Inner Siderail Panel Assembly for Smart TV (Left Shown)

Head End Siderail Assembly

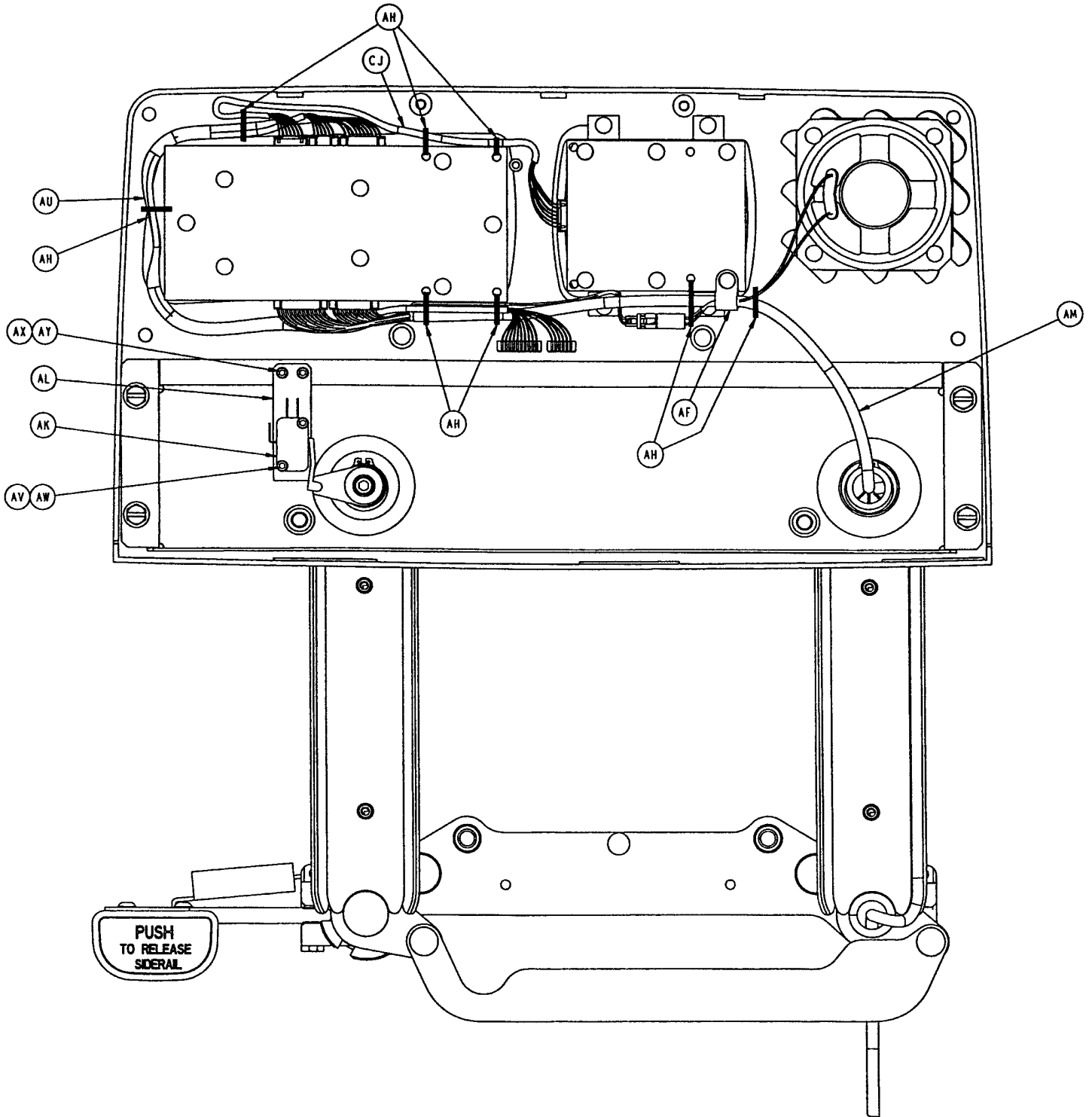

Left Head Siderail Wire Routing for Smart TV Option

Head End Siderail Assembly

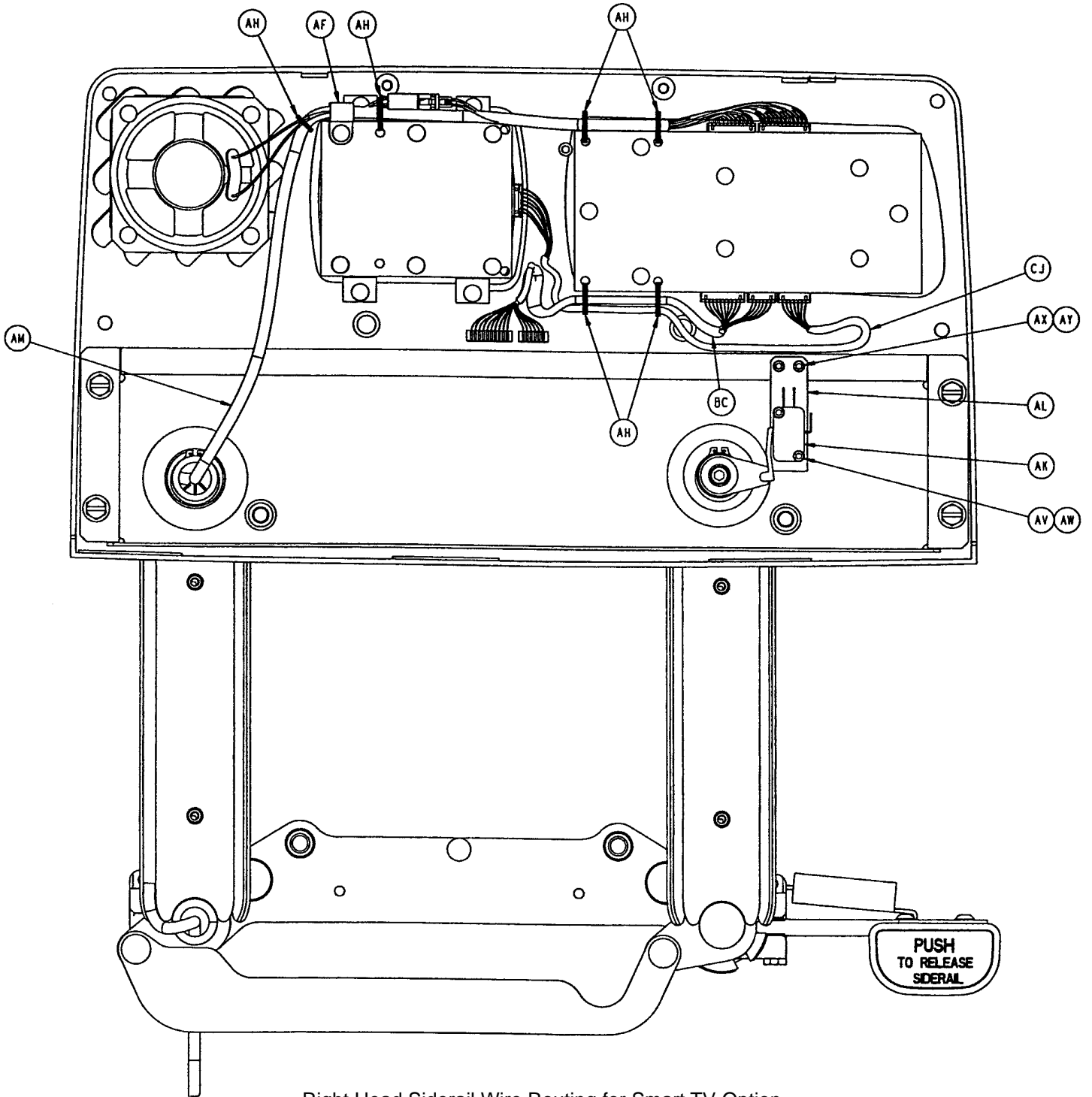

Right Head Siderail Wire Routing for Smart TV Option

Head End Siderail Assembly

2035-400-105 Left Standard Components

Item	Part No.	Part Name	Qty.
E	3001-400-513	Wear Bushing	1
F	2035-400-531	Siderail Carrier	1
G	3001-400-530	Siderail Arm Spacer	1
H	3000-400-513	Flange Bearing	7
J	3000-400-557	Sleeve Bearing	4
K	11-353	Shim Washer	18
L	28-128	Retaining Ring	2
M	2035-400-553	Release Lever, Head Rail	1
N	3000-400-556	Warning Label	1
P	23-280	High-Low Tapping Screw	4
Q	5000-20-5	Inner Arm Cover	2
R	3-344	Hex Hd. Mach. Screw	2
S	3000-200-334	Extension Spring	1
T	5000-20-6	Outer Arm Cover	2
V	2035-400-570	Glide Rod	2
W	23-90	High-Low Tapping Screw	8
X	3001-400-130	Supt. Weldment, Hd., Lt.	1
Y	11-434	Shim	2
Z	28-132	Bowed Retaining Ring	3
AA	3000-400-523	Panel Spacer	2
AB	3001-400-515	Head Rail	1
AC	3001-400-558	Siderail Spacer	4
AD	3-226	Hex Washer Hd. Screw	4
AE	23-86	High-Low Tapping Screw	1
AF	3000-300-478	CPR Conduit Clamp	1
AG	23-112	High-Low Tapping Screw	16
AH	3000-300-114	4" Cable Tie	8
AJ	2035-20-60	Limit Switch Cam	1
AK	3000-300-41	Micro Switch	1
AL	2035-20-62	Limit Switch Bracket	1
AM	2035-20-802	Siderail Cable	1
AN	2035-32-801	Limit Switch Cable	1
AP	(page 11-61)	Outer Panel Assembly	1
AQ	(page 11-59)	Timing Link Ass'y, Hd., Lt.	1
AR	2035-400-900	Siderail Outside PCB	1
AS	(page 11-60)	Inner Panel Assembly	1
AU	2035-20-804	Main Outside Cable, Lt.	1
AV	4-127	Soc. Hd. Cap Screw	2
AW	16-69	Twin Fastener	1
AX	4-101	Soc. Hd. Cap Screw	2
AY	16-23	Fiberlock Nut	2
AZ	2035-20-61	Insulation Card	1
BA	4-9	Soc. Hd. Cap Screw	1
BB	13-10	Ext. Tooth Lock Washer	1
BC	3-75	Hex Hd. Cap Screw	1
BD	4-278	Hex But. Hd. Cap Screw	2
BE	11-338	Wave Washer	6
BF	11-344	Washer	1
BG	16-2	Nylock Nut	2
BH	38-435	Wave Washer	1
BJ	3001-400-505	Release Pad Label	1
BK	3001-400-514	Release Lever Pad	1


2035-400-205 Right Standard Components

Item	Part No.	Part Name	Qty.
E	3001-400-513	Wear Bushing	1
F	2035-400-531	Siderail Carrier	1
G	3001-400-530	Siderail Arm Spacer	1
H	3000-400-513	Flange Bearing	7
J	3000-400-557	Sleeve Bearing	4
K	11-353	Shim Washer	18
L	28-128	Retaining Ring	2
M	2035-400-553	Release Lever, Head Rail	1
N	3000-400-556	Warning Label	1
P	23-280	High-Low Tapping Screw	4
Q	5000-20-5	Inner Arm Cover	2
R	3-344	Hex Hd. Mach. Screw	2
S	3000-200-334	Extension Spring	1
T	5000-20-6	Outer Arm Cover	2
V	2035-400-570	Glide Rod	2
W	23-90	High-Low Tapping Screw	8
X	3001-400-230	Supt. Weldment, Hd., Rt.	1
Y	11-434	Shim	2
Z	28-132	Bowed Retaining Ring	3
AA	3000-400-523	Panel Spacer	2
AB	3001-400-515	Head Rail	1
AC	3001-400-558	Siderail Spacer	4
AD	3-226	Hex Washer Hd. Screw	4
AE	23-86	High-Low Tapping Screw	1
AF	3000-300-478	CPR Conduit Clamp	1
AG	23-112	High-Low Tapping Screw	16
AH	3000-300-114	4" Cable Tie	7
AJ	2035-20-60	Limit Switch Cam	1
AK	3000-300-41	Micro Switch	1
AL	2035-20-62	Limit Switch Bracket	1
AM	2035-20-802	Siderail Cable	1
AN	2035-32-801	Limit Switch Cable	1
AP	(page 11-61)	Outer Panel Assembly	1
AQ	(page 11-58)	Timing Link Ass'y, Hd., Rt.	1
AR	2035-400-900	Siderail Outside PCB	1
AS	(page 11-60)	Inner Panel Assembly	1
AU	2035-20-803	Main Outside Cable, Rt.	1
AV	4-127	Soc. Hd. Cap Screw	2
AW	16-69	Twin Fastener	1
AX	4-101	Soc. Hd. Cap Screw	2
AY	16-23	Fiberlock Nut	2
AZ	2035-20-61	Insulation Card	1
BA	4-9	Soc. Hd. Cap Screw	1
BB	13-10	Ext. Tooth Lock Washer	1
BC	3-75	Hex Hd. Cap Screw	1
BD	4-278	Hex But. Hd. Cap Screw	2
BE	11-338	Wave Washer	6
BF	11-344	Washer	1
BG	16-2	Nylock Nut	2
BH	38-435	Wave Washer	1
BJ	3001-400-505	Release Pad Label	1
BK	3001-400-514	Release Lever Pad	1


Head End Siderail Assembly

2035-20-11 Standard Siderail

Item	Part No.	Part Name	Qty.
CA	2035-000-100	Label, Standard, Left	1
CA	2035-000-200	Label, Standard, Right	1
CC	3001-400-953	Switch Cap	28
CD	3001-400-522	Filler Cap	18
CE	3001-400-517	Speaker Seal	2
CF	3001-400-535	Inner Panel Blank Module	2
CH	2035-000-300	Label, Standard, Left	1
CH	2035-000-400	Label, Standard, Right	1

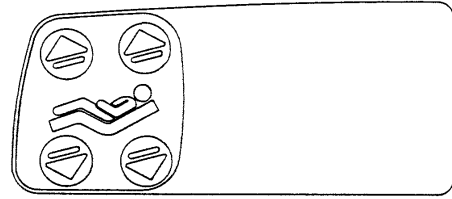


Right Inner Siderail Label

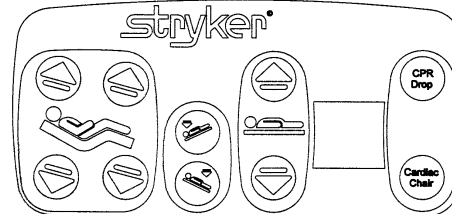


Right Outer Siderail Label

2035-20-12 Standard Siderail with Nurse Call

Item	Part No.	Part Name	Qty.
CA	2035-000-101	Label, Standard, NC, Left	1
CA	2035-000-201	Label, Standard, NC, Right	1
CC	3001-400-953	Switch Cap	32
CD	3001-400-522	Filler Cap	14
CE	3001-403-831	Speaker with Cable	2
CF	3001-400-535	Inner Panel Blank Module	2
CH	2035-000-301	Label, Standard, NC, Left	1
CH	2035-000-401	Label, Standard, NC, Right	1

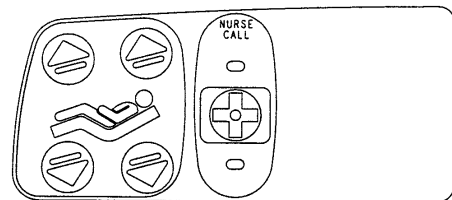


Right Inner Siderail Label

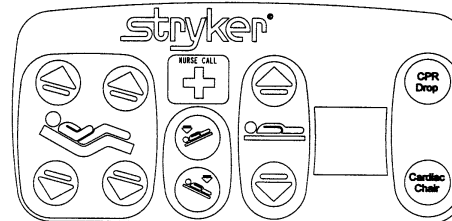


Right Outer Siderail Label

2035-20-16 Standard Siderail with DMS


Item	Part No.	Part Name	Qty.
AN	3000-300-114	Cable Tie	3
CA	2035-000-100	Label, Standard, Left	1
CA	2035-000-200	Label, Standard, Right	1
CB	2035-000-104	Label, Mattress, Left	1
CB	2035-000-204	Label, Mattress, Right	1
CC	3001-400-953	Switch Cap	36
CD	3001-400-522	Filler Cap	14
CE	3001-400-517	Speaker Seal	2
CF	(page 11-62)	DMS Module Assembly	2
CH	2035-000-303	Label, Standard, DMS, Lt.	1
CH	2035-000-403	Label, Standard, DMS, Rt.	1
CJ	3001-402-803	Main to Opt. PCB Cable, Rt.	1
CJ	3001-402-804	Main to Opt. PCB Cable, Lt.	1

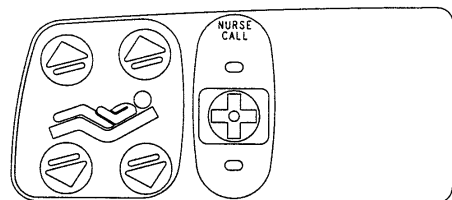

Right Inner Siderail Label

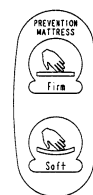

Inner Mattress Label

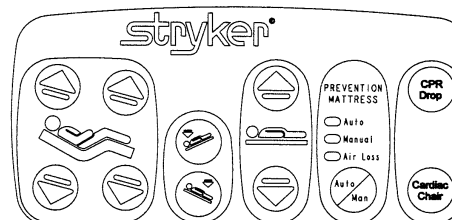


Right Outer Siderail Label


Head End Siderail Assembly

2035-20-19 Standard Siderail with StryKair


Item	Part No.	Part Name	Qty.
AN	3000-300-114	Cable Tie	3
CA	2035-000-100	Label, Standard, Left	1
CA	2035-000-200	Label, Standard, Right	1
CB	2035-000-104	Label, Mattress, Left	1
CB	2035-000-204	Label, Mattress, Right	1
CC	3001-400-953	Switch Cap	36
CD	3001-400-522	Filler Cap	14
CE	3001-400-517	Speaker Seal	2
CF	(page 11-62)	DMS Module Assembly	2
CH	2035-000-305	Label, Std., StryKair, Lt.	1
CH	2035-000-405	Label, Std., StryKair, Rt.	1
CJ	3001-402-803	Main to Opt. PCB Cable, Rt.	1
CJ	3001-402-804	Main to Opt. PCB Cable, Lt.	1

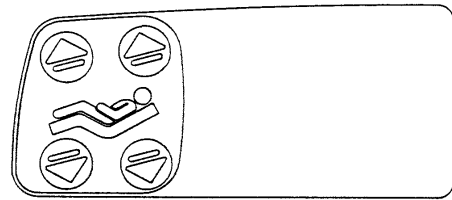

Right Inner Siderail Label

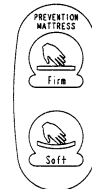


Inner Mattress Label

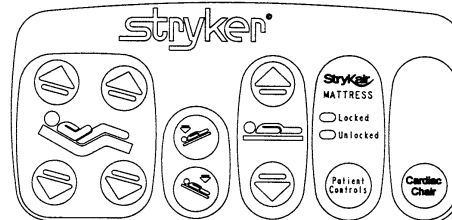


Right Outer Siderail Label

2035-20-32 Std. Siderail/StryKair Capable

Item	Part No.	Part Name	Qty.
AN	3000-300-114	Cable Tie	3
CA	2035-000-100	Label, Standard, Left	1
CA	2035-000-200	Label, Standard, Right	1
CB	2035-000-105	Label, Blank	2
CC	3001-400-953	Switch Cap	32
CD	3001-400-522	Filler Cap	18
CE	3001-400-517	Speaker Seal	2
CF	(page 11-62)	DMS Module Assembly	2
CH	2035-000-307	Label, StryKair Capable, Lt.	1
CH	2035-000-407	Label, StryKair Capable, Rt.	1
CJ	3001-402-803	Main to Opt. PCB Cable, Rt.	1
CJ	3001-402-804	Main to Opt. PCB Cable, Lt.	1

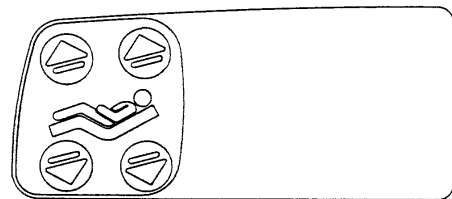


Right Inner Siderail Label

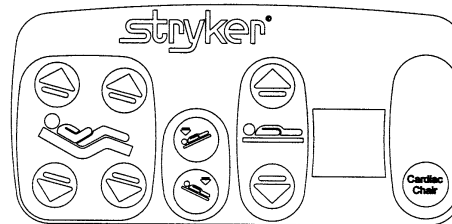


Right Outer Siderail Label

2035-20-13 Standard Siderail w/NC & DMS


Item	Part No.	Part Name	Qty.
AN	3000-300-114	Cable Tie	3
CA	2035-000-101	Label, Standard, NC, Left	1
CA	2035-000-201	Label, Standard, NC, Right	1
CB	2035-000-104	Label, Mattress, Left	1
CB	2035-000-204	Label, Mattress, Right	1
CC	3001-400-953	Switch Cap	38
CD	3001-400-522	Filler Cap	12
CE	3001-403-831	Speaker with Cable	2
CF	(page 11-62)	DMS Module Assembly	2
CH	2035-000-302	Label, Std., NC, DMS, Lt.	1
CH	2035-000-402	Label, Std., NC, DMS, Rt.	1
CJ	3001-402-803	Main to Opt. PCB Cable, Rt.	1
CJ	3001-402-804	Main to Opt. PCB Cable, Lt.	1

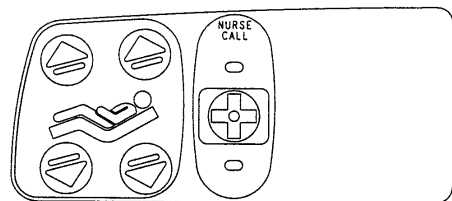

Right Inner Siderail Label

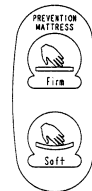

Inner Mattress Label

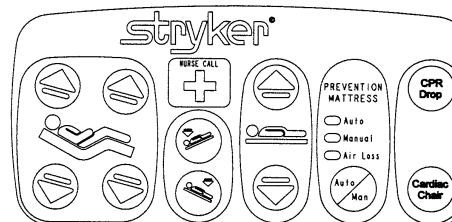


Right Outer Siderail Label


Head End Siderail Assembly

2035-20-17 Std. Siderail w/NC & StryKair


Item	Part No.	Part Name	Qty.
AN	3000-300-114	Cable Tie	3
CA	2035-000-101	Label, Standard, NC, Left	1
CA	2035-000-201	Label, Standard, NC, Right	1
CB	2035-000-104	Label, Mattress, Left	1
CB	2035-000-204	Label, Mattress, Right	1
CC	3001-400-953	Switch Cap	40
CD	3001-400-522	Filler Cap	10
CE	3001-403-831	Speaker with Cable	2
CF	(page 11-62)	DMS Module Assembly	2
CH	2035-000-304	Label, Std., NC, StryKair, Lt.	1
CH	2035-000-404	Label, Std., NC, StryKair, Rt.	1
CJ	3001-402-803	Main to Opt. PCB Cable, Rt.	1
CJ	3001-402-804	Main to Opt. PCB Cable, Lt.	1

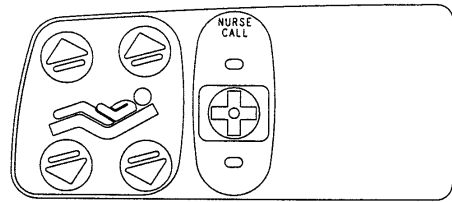

Right Inner Siderail Label

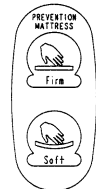


Inner Mattress Label

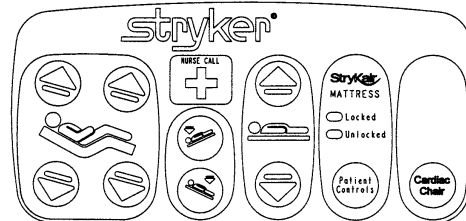


Right Outer Siderail Label

2035-20-30 Std. Rail w/NC/Strykair Capable

Item	Part No.	Part Name	Qty.
AN	3000-300-114	Cable Tie	3
CA	2035-000-101	Label, Standard, NC, Left	1
CA	2035-000-201	Label, Standard, NC, Right	1
CB	2035-000-105	Blank Label	2
CC	3001-400-953	Switch Cap	38
CD	3001-400-522	Filler Cap	12
CE	3001-403-831	Speaker with Cable	2
CF	(page 11-62)	DMS Module Assembly	2
CH	2035-000-306	Label, NC, StryKair, Cap. Lt.	1
CH	2035-000-406	Label, NC, StryKair, Cap. Rt.	1
CJ	3001-402-803	Main to Opt. PCB Cable, Rt.	1
CJ	3001-402-804	Main to Opt. PCB Cable, Lt.	1

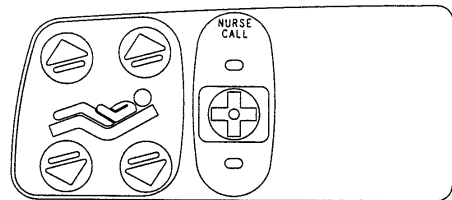


Right Inner Siderail Label

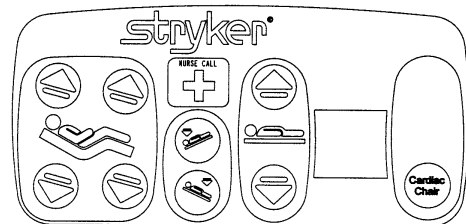


Right Outer Siderail Label

2035-20-15 Standard Siderail w/NC & Comm.

Item	Part No.	Part Name	Qty.
CA	2035-000-102	Label, Standard, Left	1
CA	2035-000-202	Label, Standard, Right	1
CC	3001-400-953	Switch Cap	42
CD	3001-400-522	Filler Cap	4
CE	3001-403-831	Speaker with Cable	2
CF	3001-400-535	Inner Panel Blank Module	2
CH	2035-000-301	Label, Standard, NC, Lt.	1
CH	2035-000-401	Label, Standard, NC, Rt.	1

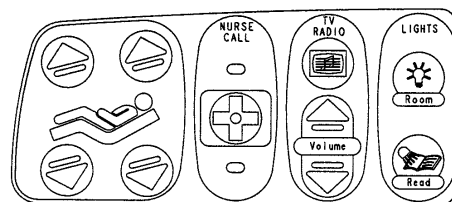

Right Inner Siderail Label

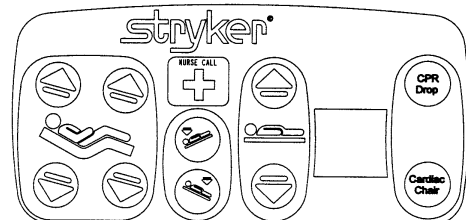


Right Outer Siderail Label


Head End Siderail Assembly

2035-20-21 Standard Siderail w/NC & STV


Item	Part No.	Part Name	Qty.
AN	3000-300-114	Cable Tie	3
CA	2035-000-106	Label, Std., NC, STV., Lt.	1
CA	2035-000-206	Label, Std., NC, STV., Rt.	1
CB	2035-000-109	Label, CC & Mute, Left	1
CB	2035-000-209	Label, CC & Mute, Right	1
CC	3001-400-953	Switch Cap	42
CD	3001-400-522	Filler Cap	4
CE	3001-403-831	Speaker with Cable	2
CF	(page 11-63)	STV Module, Left	1
CF	(page 11-64)	STV Module, Right	1
CH	2035-000-301	Label, Standard, NC, Lt.	1
CH	2035-000-401	Label, Standard, NC, Rt.	1
CJ	3001-402-803	Main to Opt. PCB Cable	2

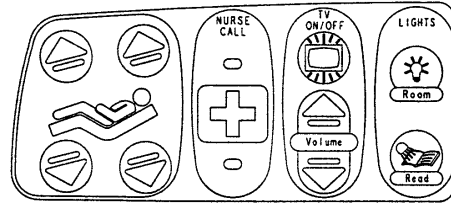

Right Inner Siderail Label

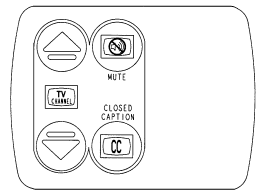


Inner STV Label

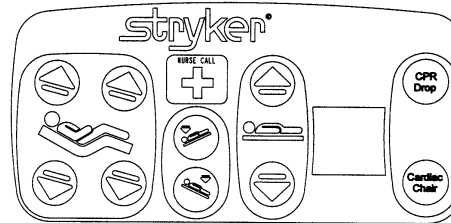


Right Outer Siderail Label

2035-20-14 Std. Rail w/NC, DMS & Comm.


Item	Part No.	Part Name	Qty.
AN	3000-300-114	Cable Tie	3
CA	2035-000-102	Label, Std., NC, Comm., Lt.	1
CA	2035-000-202	Label, St., NC, Comm. Rt.	1
CB	2035-000-104	Label, Mattress, Left	1
CB	2035-000-204	Label, Mattress, Right	1
CC	3001-400-953	Switch Cap	48
CD	3001-400-522	Filler Cap	2
CE	3001-403-831	Speaker with Cable	2
CF	(page 11-62)	DMS Module Assembly	2
CH	2035-000-302	Label, Std., NC, DMS, Lt.	1
CH	2035-000-402	Label, Std., NC, DMS, Rt.	1
CJ	3001-402-803	Main to Opt. PCB Cable, Rt.	1
CJ	3001-402-804	Main to Opt. PCB Cable, Lt.	1

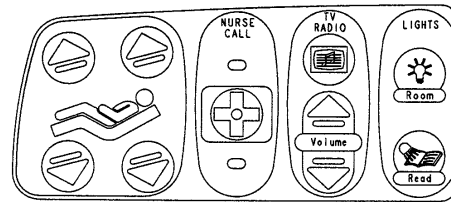

Right Inner Siderail Label

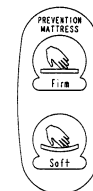


Inner Mattress Label

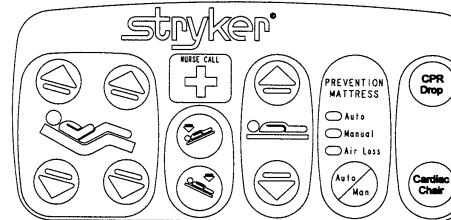


Right Outer Siderail Label

2035-20-20 Std. Siderail w/NC, DMS & STV


Item	Part No.	Part Name	Qty.
AN	3000-300-114	Cable Tie	3
CA	2035-000-106	Label, Std., NC, STV., Lt.	1
CA	2035-000-206	Label, Std., NC, STV., Rt.	1
CB	2035-000-110	Label, Matt., CC & Mute, Lt.	1
CB	2035-000-210	Label, Matt., CC & Mute, Rt.	1
CC	3001-400-953	Switch Cap	48
CD	3001-400-522	Filler Cap	2
CE	3001-403-831	Speaker with Cable	2
CF	(page 11-65)	Module Assembly, Lt.	1
CF	(page 11-66)	Module Assembly, Rt.	1
CH	2035-000-302	Label, Standard, NC, Lt.	1
CH	2035-000-402	Label, Standard, NC, Rt.	1
CJ	3001-402-803	Main to Opt. PCB Cable	2

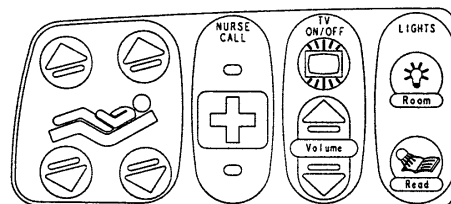

Right Inner Siderail Label

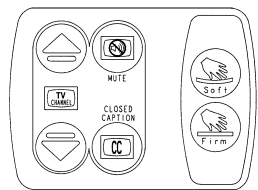

Inner Mattress/STV Label

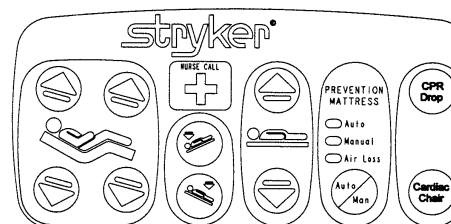


Right Outer Siderail Label

Head End Siderail Assembly


2035-20-18 Std. Rail w/NC, StryKair & Comm.

Item	Part No.	Part Name	Qty.
AN	3000-300-114	Cable Tie	3
CA	2035-000-102	Label, Std., NC, Comm., Lt.	1
CA	2035-000-202	Label, Std., NC, Comm. Rt.	1
CB	2035-000-104	Label, Mattress, Left	1
CB	2035-000-204	Label, Mattress, Right	1
CC	3001-400-953	Switch Cap	50
CE	3001-403-831	Speaker with Cable	2
CF	(page 11-62)	DMS Module Assembly	2
CH	2035-000-304	Label, Std., NC, StryKair, Lt.	1
CH	2035-000-404	Label, Std., NC, StryKair, Rt.	1
CJ	3001-402-803	Main to Opt. PCB Cable, Rt.	1
CJ	3001-402-804	Main to Opt. PCB Cable, Lt.	1

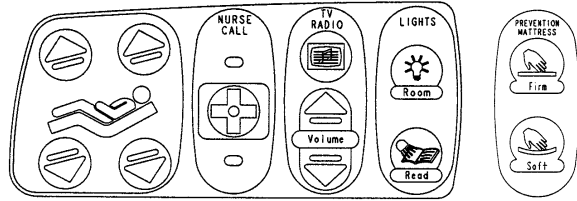

Right Inner Siderail Label


Inner Mattress Label

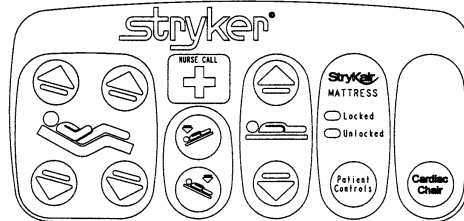


Right Outer Siderail Label

2035-20-31 Rail w/NC, StryKair Cap. & Comm.

Item	Part No.	Part Name	Qty.
AN	3000-300-114	Cable Tie	3
CA	2035-000-102	Label, Std., NC, Comm. Lt.	1
CA	2035-000-202	Label, Std., NC, Comm. Rt.	1
CB	2035-000-105	Blank Label	2
CC	3001-400-953	Switch Cap	46
CD	3001-400-522	Filler Cap	4
CE	3001-403-831	Speaker with Cable	2
CF	(page 11-62)	DMS Module Assembly	2
CH	2035-000-306	Label, NC, StryKair, Cap. Lt.	1
CH	2035-000-406	Label, NC, StryKair, Cap. Rt.	1
CJ	3001-402-803	Main to Opt. PCB Cable, Rt.	1
CJ	3001-402-804	Main to Opt. PCB Cable, Lt.	1

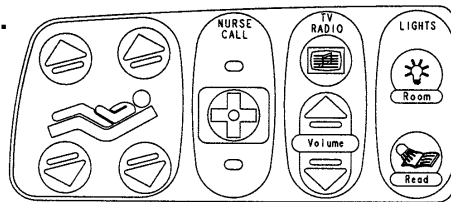


Right Inner Siderail Label

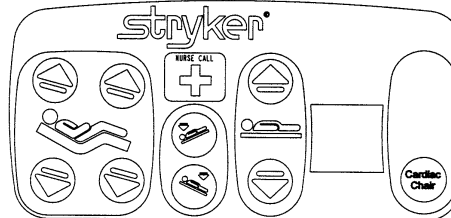


Right Outer Siderail Label

2035-20-22 Std. Rail w/NC, StryKair & STV


Item	Part No.	Part Name	Qty.
AN	3000-300-114	Cable Tie	3
CA	2035-000-106	Label, Std., NC, STV., Lt.	1
CA	2035-000-206	Label, Std., NC, STV., Rt.	1
CB	2035-000-110	Label, Matt., CC & Mute, Lt.	1
CB	2035-000-210	Label, Matt., CC & Mute, Rt.	1
CC	3001-400-953	Switch Cap	48
CD	3001-400-522	Filler Cap	2
CE	3001-403-831	Speaker with Cable	2
CF	(page 11-65)	Module Assembly, Lt.	1
CF	(page 11-66)	Module Assembly, Rt.	1
CH	2035-000-304	Label, Std., NC, StryKair, Lt.	1
CH	2035-000-404	Label, Std., NC, StryKair, Rt.	1
CJ	3001-402-803	Main to Opt. PCB Cable	2

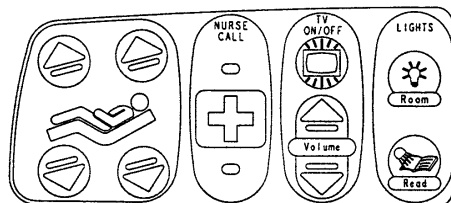

Right Inner Siderail Label

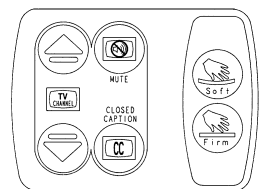


Inner Mattress/STV Label

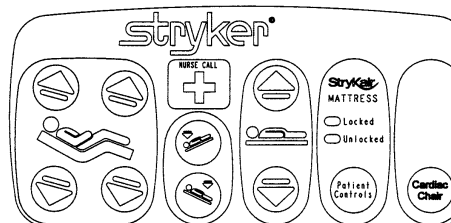


Right Outer Siderail Label

2035-400-228 Head End Siderail Timing Link Assembly, Right

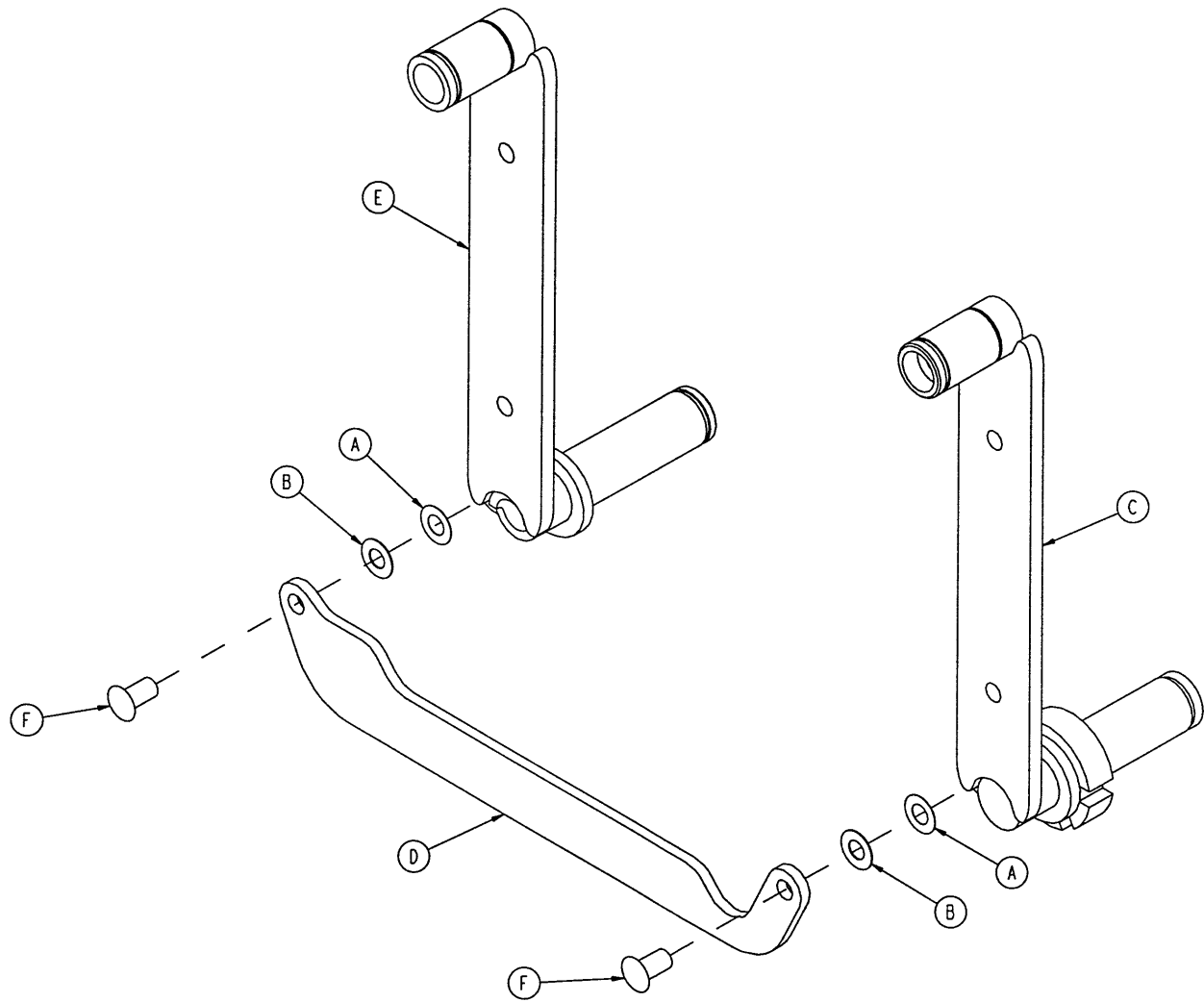


Item	Part No.	Part Name	Qty.
A	11-377	Nylon Washer	2
B	11-403	Shim Washer	2
C	2035-400-227	Arm Wldmt., Right, Head, Foot	1
D	3001-400-11	Head End Timing Link	1
E	3001-400-228	Arm Wldmt., Right, Head, Head	1
F	3001-400-501	Siderail Linkage Rivet	2

2035-400-128 Head End Siderail Timing Link Assembly, Left

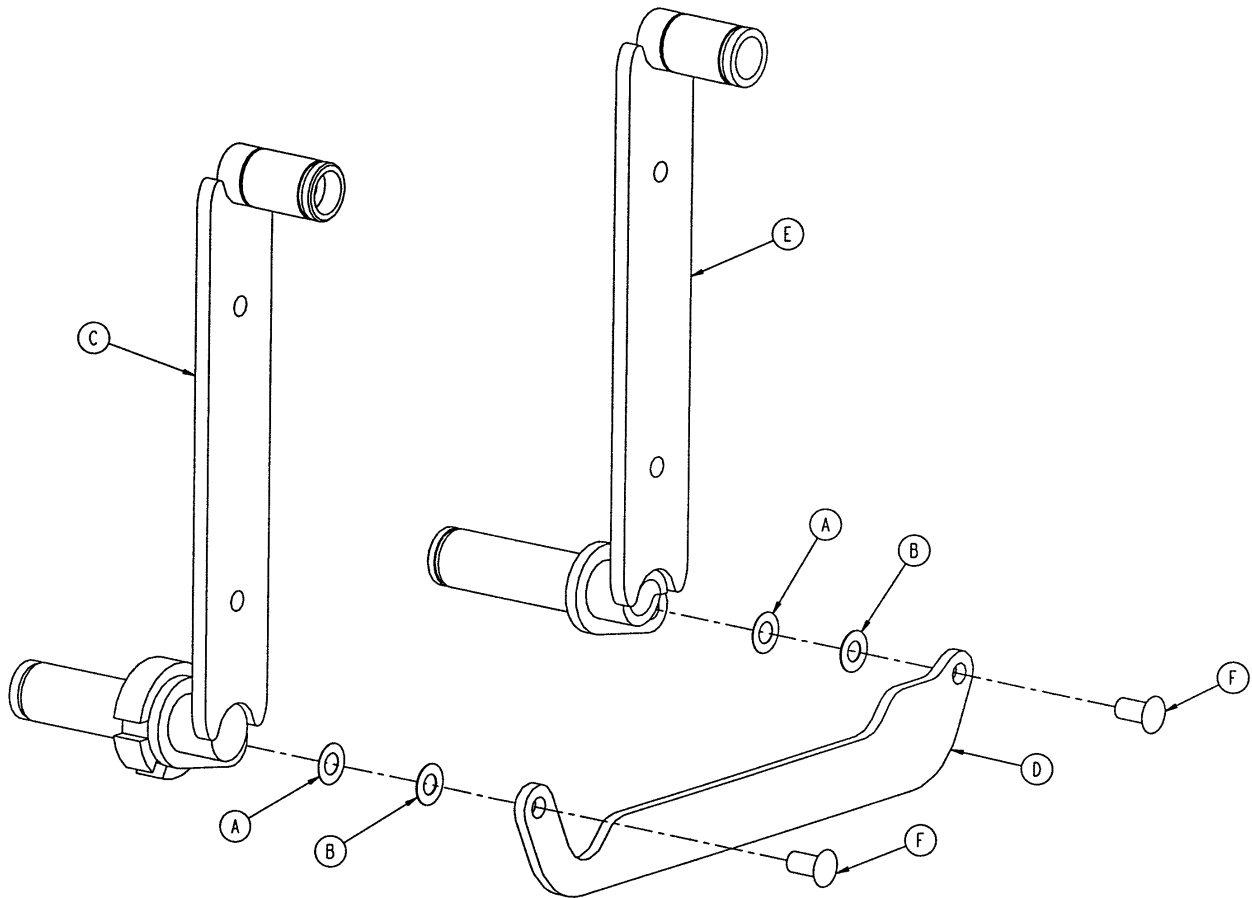

Item	Part No.	Part Name	Qty.
A	11-377	Nylon Washer	2
B	11-403	Shim Washer	2
C	2035-400-127	Arm Wldmt., Left, Head, Foot	1
D	3001-400-11	Head End Timing Link	1
E	3001-400-128	Arm Wldmt., Left, Head, Head	1
F	3001-400-501	Siderail Linkage Rivet	2


Head End Siderail Inner Panel Assembly

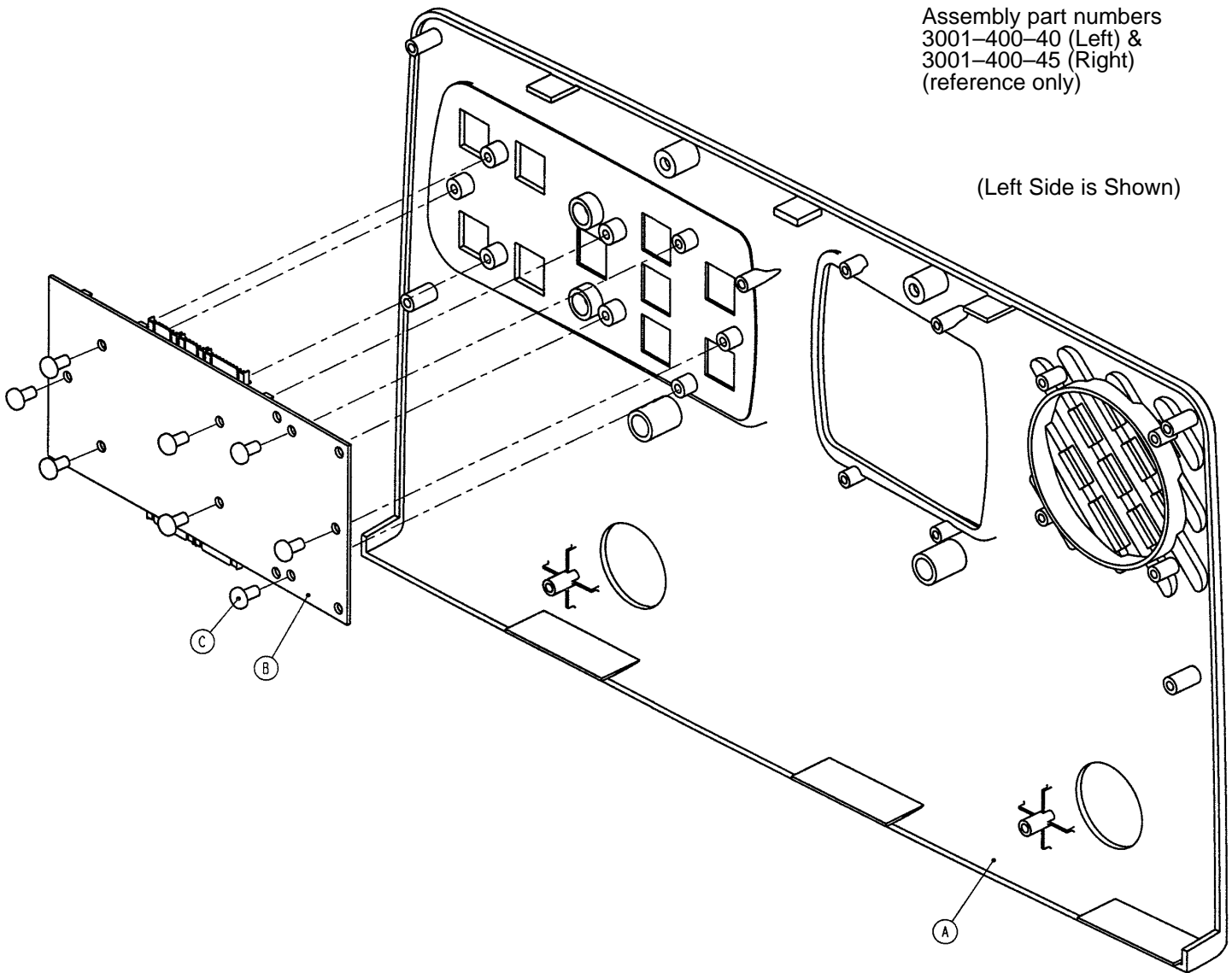

Item	Part No.	Part Name	Qty.
A	3001-400-101	Left Inner Panel	1
	3001-400-201	Right Inner Panel	1
B	3001-400-900	Inner Siderail PCB Assembly	1
C	23-112	Hi-Low Tapping Screw	8


Head End Siderail Outer Panel Assembly, Right & Left

Assembly part number 2035-400-50
(reference only)

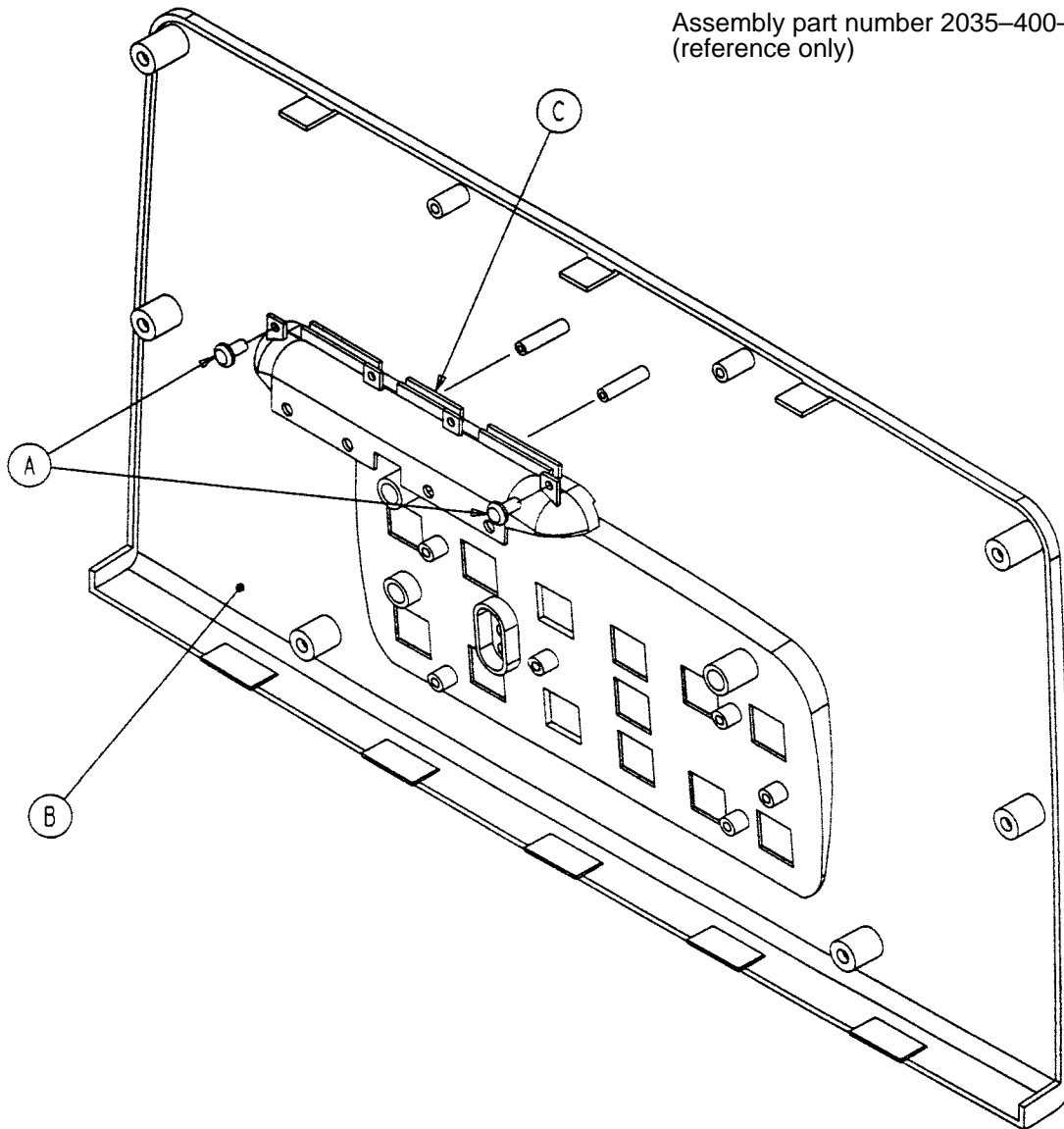


Item	Part No.	Part Name	Qty.
A	23-112	Hi-Low Tapping Screw	2
B	2035-400-102	Outer Panel	1
C	3001-400-599	Handle Insert	1

Optional Siderail DMS Module Assembly, Right & Left

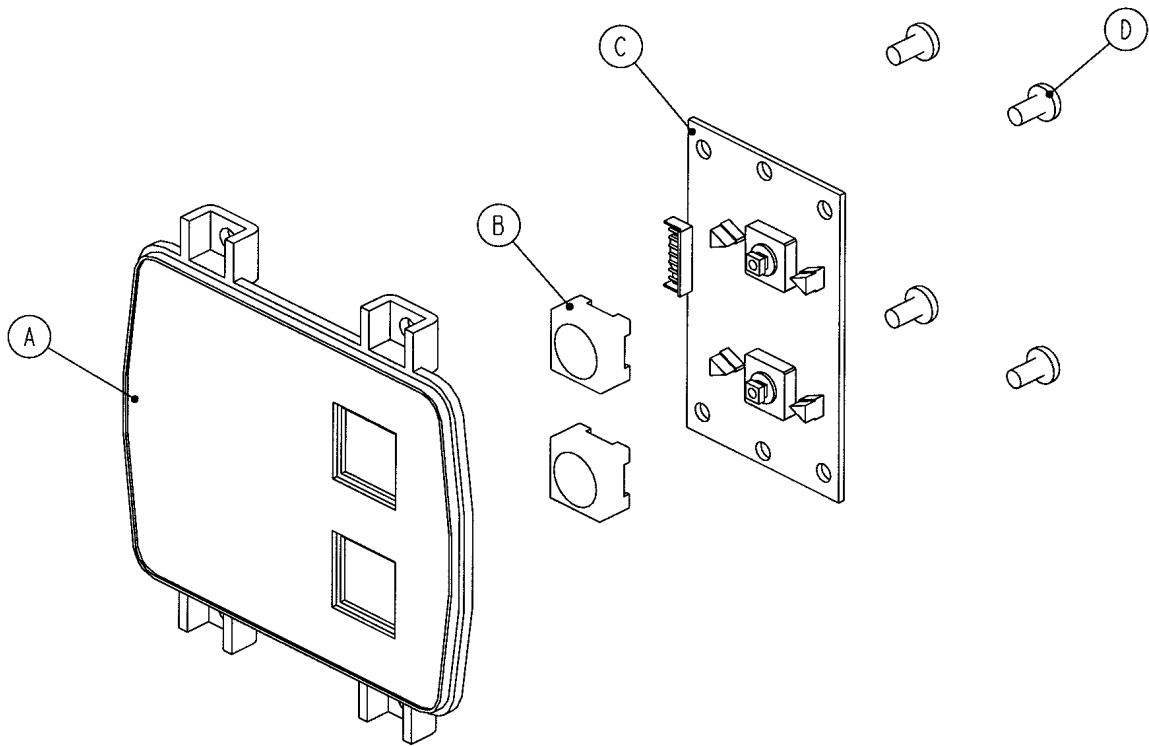


Item	Part No.	Part Name	Qty.
A	3001-400-521	DMS Module	1
B	3001-400-953	Switch Cap	2
C	3001-402-900	DMS Keypad PCB	1
D	23-112	Tapping Screw	4

2035-400-53 Optional Smart TV Siderail Module, Right

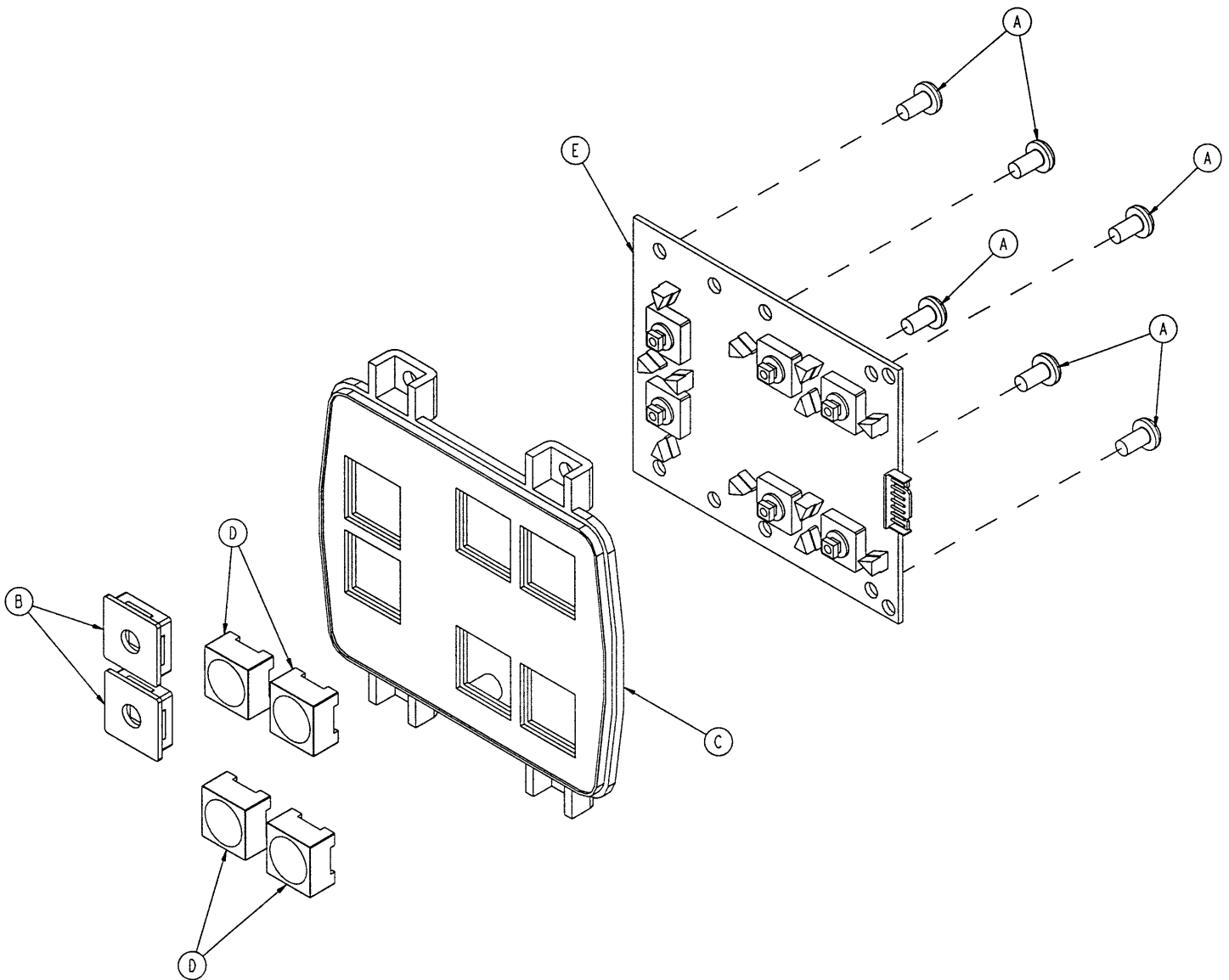


Item	Part No.	Part Name	Qty.
A	23-112	Ph. Hd. Hi-Lo Tapping Screw	6
B	3001-400-522	Filler Cap	2
C	3001-400-524	Module	1
D	3001-400-953	Switch Cap	4
E	5000-400-930	Keypad PCB, Left	1

2035–400–51 Optional Smart TV Siderail Module, Left

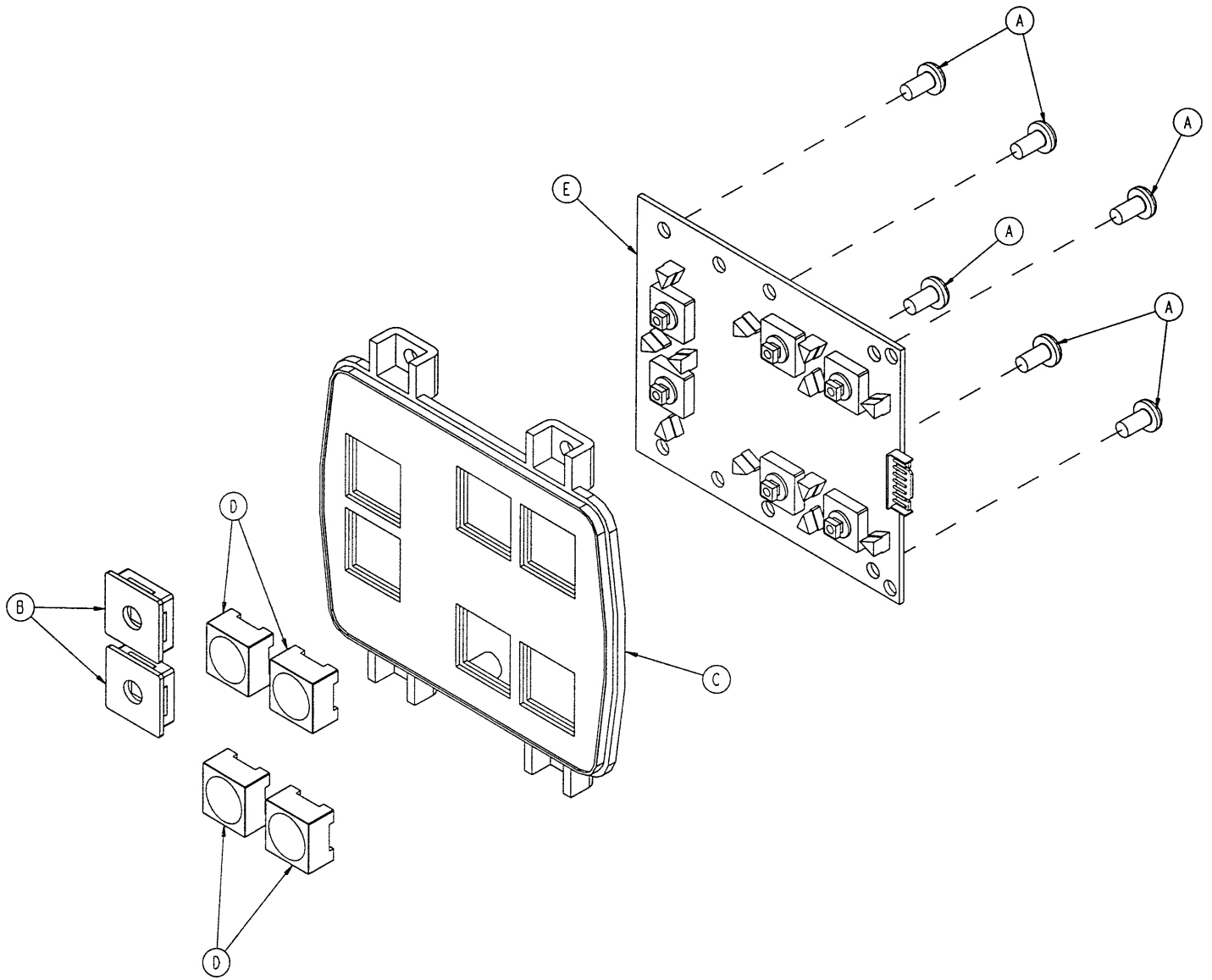


Item	Part No.	Part Name	Qty.
A	23-112	Ph. Hd. Hi-Lo Tapping Screw	6
B	3001-400-522	Filler Cap	2
C	3001-400-524	Module	1
D	3001-400-953	Switch Cap	4
E	5000-400-920	Keypad PCB, Right	1

5000–20–27 Optional Smart TV & Mattress Control Module, Right

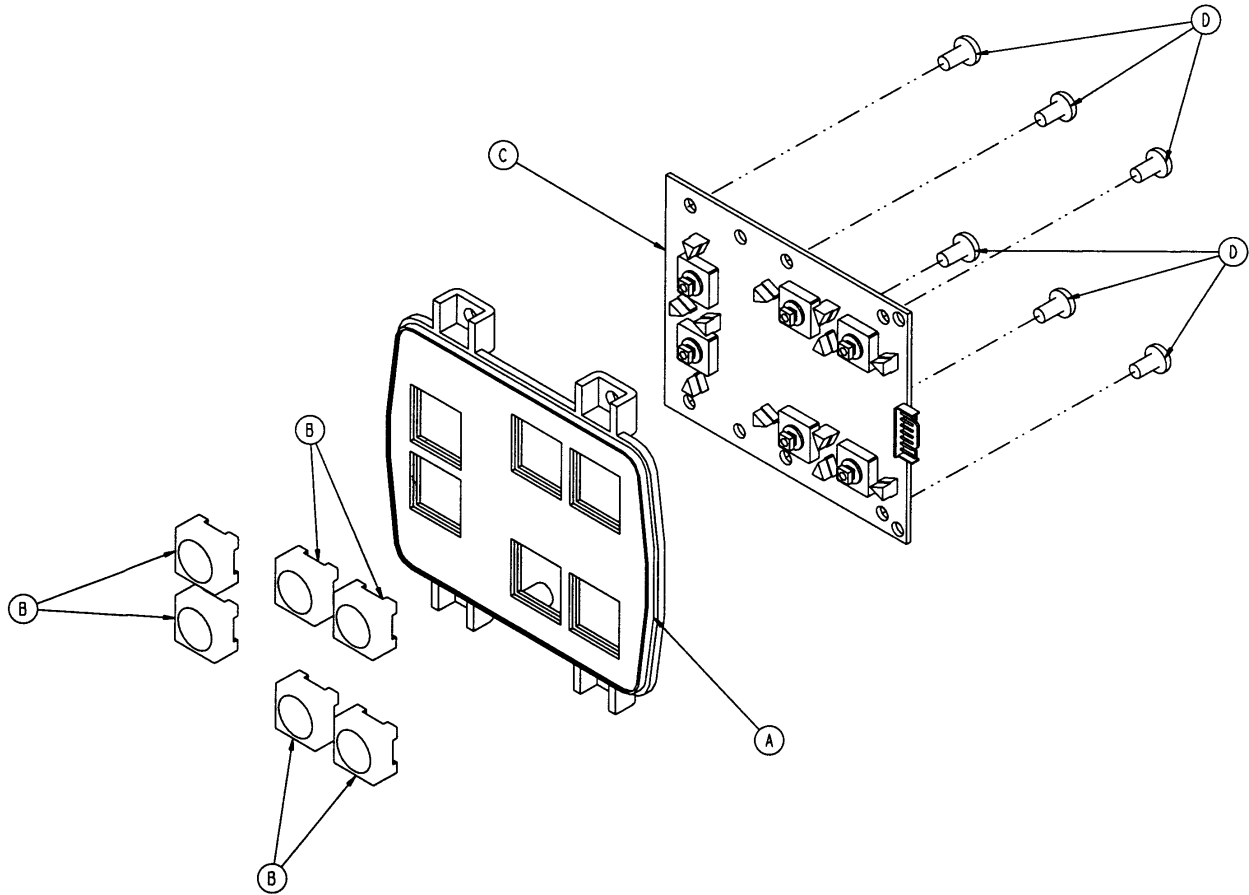

Item	Part No.	Part Name	Qty.
A	3001–400–524	Lumbar Module	1
B	3001–400–953	Switch Cap	6
C	5000–400–930	Lumbar Keypad PCB, Left	1
D	23–112	Tapping Screw	6


5000–20–26 Optional Smart TV & Mattress Control Module, Left

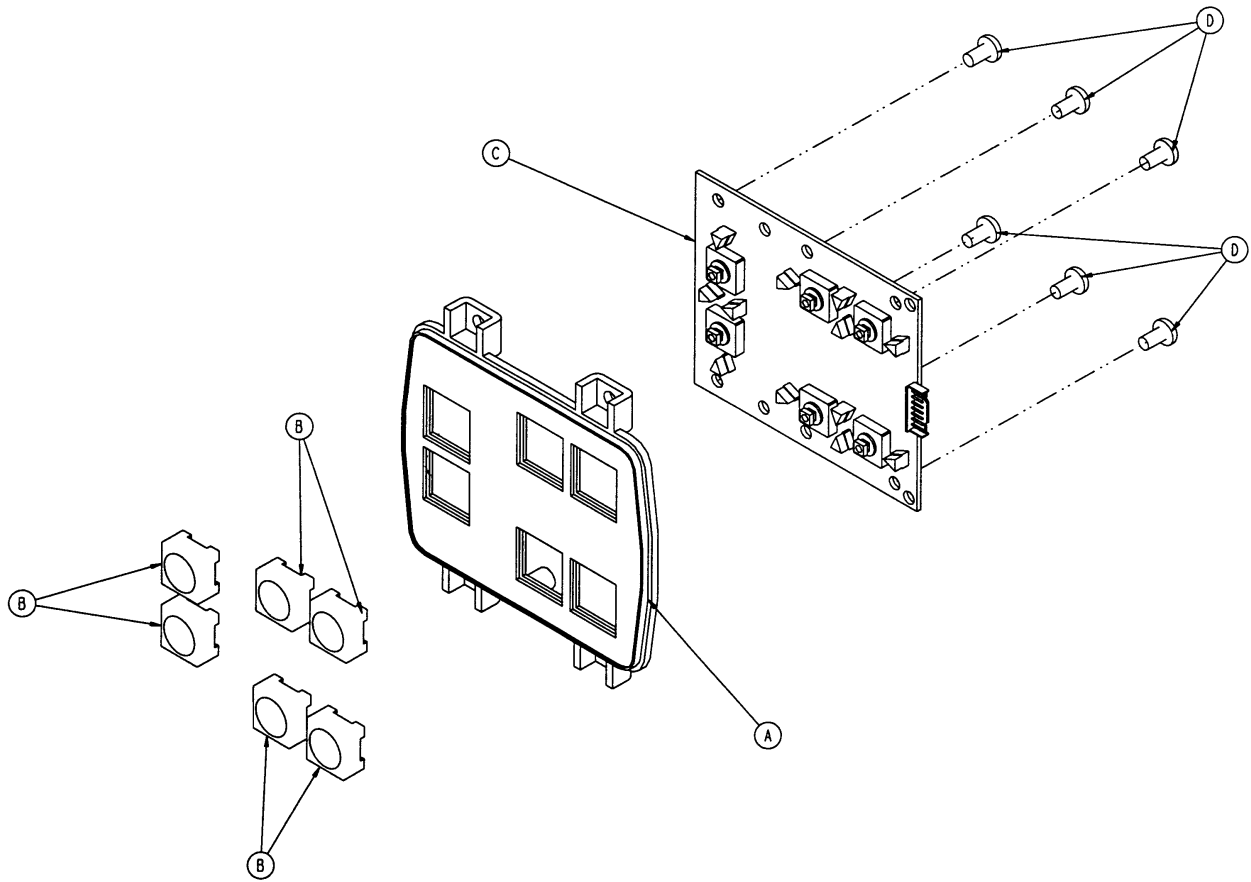


Item	Part No.	Part Name	Qty.
A	3001-400-524	Lumbar Module	1
B	3001-400-953	Switch Cap	6
C	5000-400-920	Lumbar Keypad PCB, Right	1
D	23-112	Tapping Screw	6

Foot End Siderail Assembly

Assembly part number 2035-400-405 (Left)
Assembly part number 2035-400-305 (Right)
(Reference Only)
(Right Side Shown)

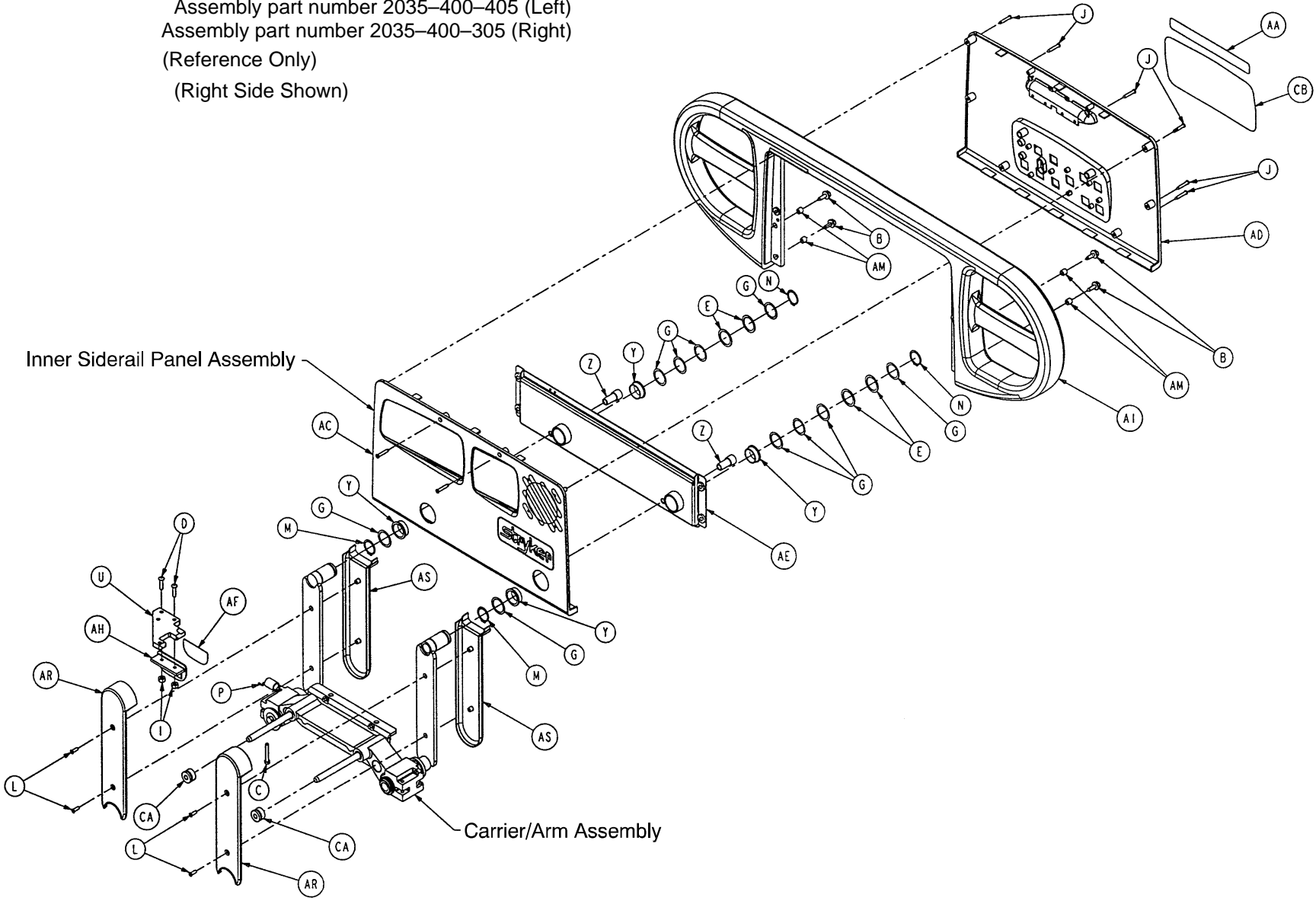


Foot End Siderail Assembly

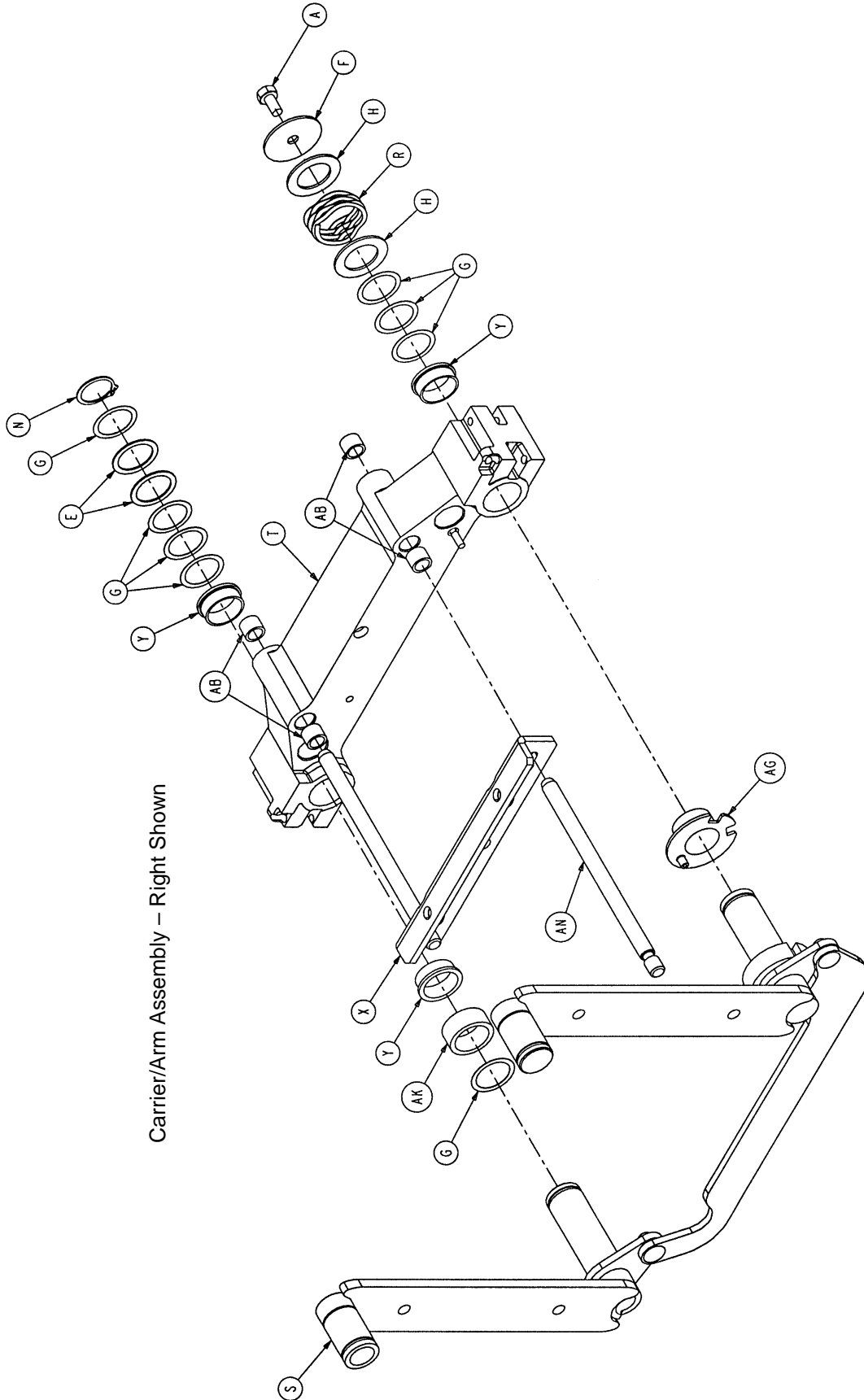

Carrier/Arm Assembly – Right Shown

Foot End Siderail Assembly

Inner Panel Assembly – Right Shown

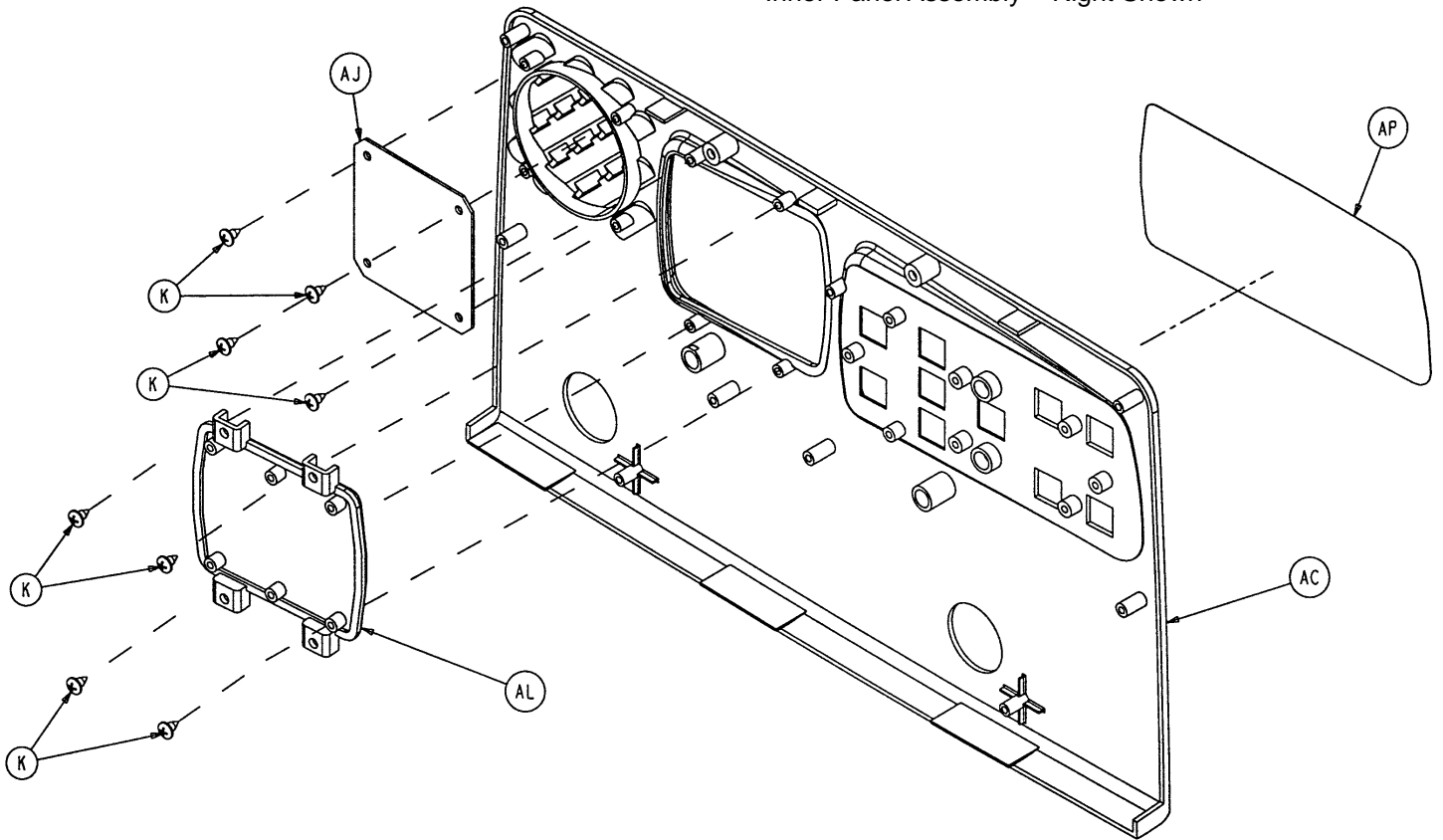

Foot End Siderail Assembly


2035-400-305 Right Siderail Components

Item	Part No.	Part Name	Qty.
A	3-75	Hex Hd. Cap Screw	1
B	3-226	Hex Washer Hd. Screw	4
C	3-330	Hex Hd. Cap Screw	1
D	4-278	Button Hd. Screw	2
E	11-338	Wave Washer	6
F	11-344	Washer	1
G	11-353	Washer	18
H	11-434	Washer	2
I	16-2	Nylock Nut	2
J	23-90	Pan Hd. Tapping Screw	8
K	23-112	Pan Hd. Tapping Screw	8
L	23-261	Flat Hd. Screw	4
M	28-128	Retaining Ring	2
N	28-132	Bowed Retaining Ring	3
P	38-433	Extension Spring	1
R	38-435	Wave Washer	1
S	(page 11-71)	Timing Link Ass'y, Right	1
T	2035-400-531	Siderail Carrier	1
U	2035-400-552	Release Lever	1
X	2035-400-556	Glide Rod Assembly	1
Y	3000-400-513	Flange Bearing	7
Z	3000-400-523	Panel Spacer	2
AA	3000-400-556	Warning Label	1
AB	3000-400-557	Sleeve Bearing	4
AC	3001-400-201	Inner Panel, Right	1
AD	3001-400-50	Outer Panel	1
AE	3001-400-230	Head, Right Supt. Wldmt.	1
AF	3001-400-505	Pad Label	1
AG	3001-400-513	Wear Bushing	1
AH	3001-400-514	Release Lever Pad	1
AI	3001-400-515	Head Rail	1
AJ	3001-400-517	Speaker Seal	1
AK	3001-400-530	Siderail Arm Spacer	1
AL	3001-400-535	Inner Panel Blank Module	1
AM	3001-400-558	Siderail Spacer	4
AN	3001-400-564	Glide Rod	1
AP	3001-445-611	Blank Label, Right	1
AR	5000-20-5	Inner Arm Cover	2
AS	5000-20-6	Outer Arm Cover	2
CA	30-40	Grommet	2
CB	2035-000-90	Apex Logo Label	1

2035-400-405 Left Siderail Components


Item	Part No.	Part Name	Qty.
A	3-75	Hex Hd. Cap Screw	1
B	3-226	Hex Washer Hd. Screw	4
C	3-330	Hex Hd. Cap Screw	1
D	4-278	Button Hd. Screw	2
E	11-338	Wave Washer	6
F	11-344	Washer	1
G	11-353	Washer	18
H	11-434	Washer	2
I	16-2	Nylock Nut	2
J	23-90	Pan Hd. Tapping Screw	8
K	23-112	Pan Hd. Tapping Screw	8
L	23-261	Flat Hd. Screw	4
M	28-128	Retaining Ring	2
N	28-132	Bowed Retaining Ring	3
P	38-433	Extension Spring	1
R	38-435	Wave Washer	1
S	(page 11-72)	Timing Link Ass'y, Left	1
T	2035-400-531	Siderail Carrier	1
U	2035-400-552	Release Lever	1
X	2035-400-556	Glide Rod Assembly	1
Y	3000-400-513	Flange Bearing	7
Z	3000-400-523	Panel Spacer	2
AA	3000-400-556	Warning Label	1
AB	3000-400-557	Sleeve Bearing	4
AC	3001-400-101	Inner Panel, Left	1
AD	3001-400-50	Outer Panel	1
AE	3001-400-130	Head, Left Supt. Wldmt.	1
AF	3001-400-505	Pad Label	1
AG	3001-400-513	Wear Bushing	1
AH	3001-400-514	Release Lever Pad	1
AI	3001-400-515	Head Rail	1
AJ	3001-400-517	Speaker Seal	1
AK	3001-400-530	Siderail Arm Spacer	1
AL	3001-400-535	Inner Panel Blank Module	1
AM	3001-400-558	Siderail Spacer	4
AN	3001-400-564	Glide Rod	1
AP	3001-445-621	Blank Label, Left	1
AR	5000-20-5	Inner Arm Cover	2
AS	5000-20-6	Outer Arm Cover	2
CA	30-40	Grommet	2
CB	2035-000-90	Apex Logo Label	1

2035-400-428 Foot End Siderail Timing Link Ass'y, Rt.

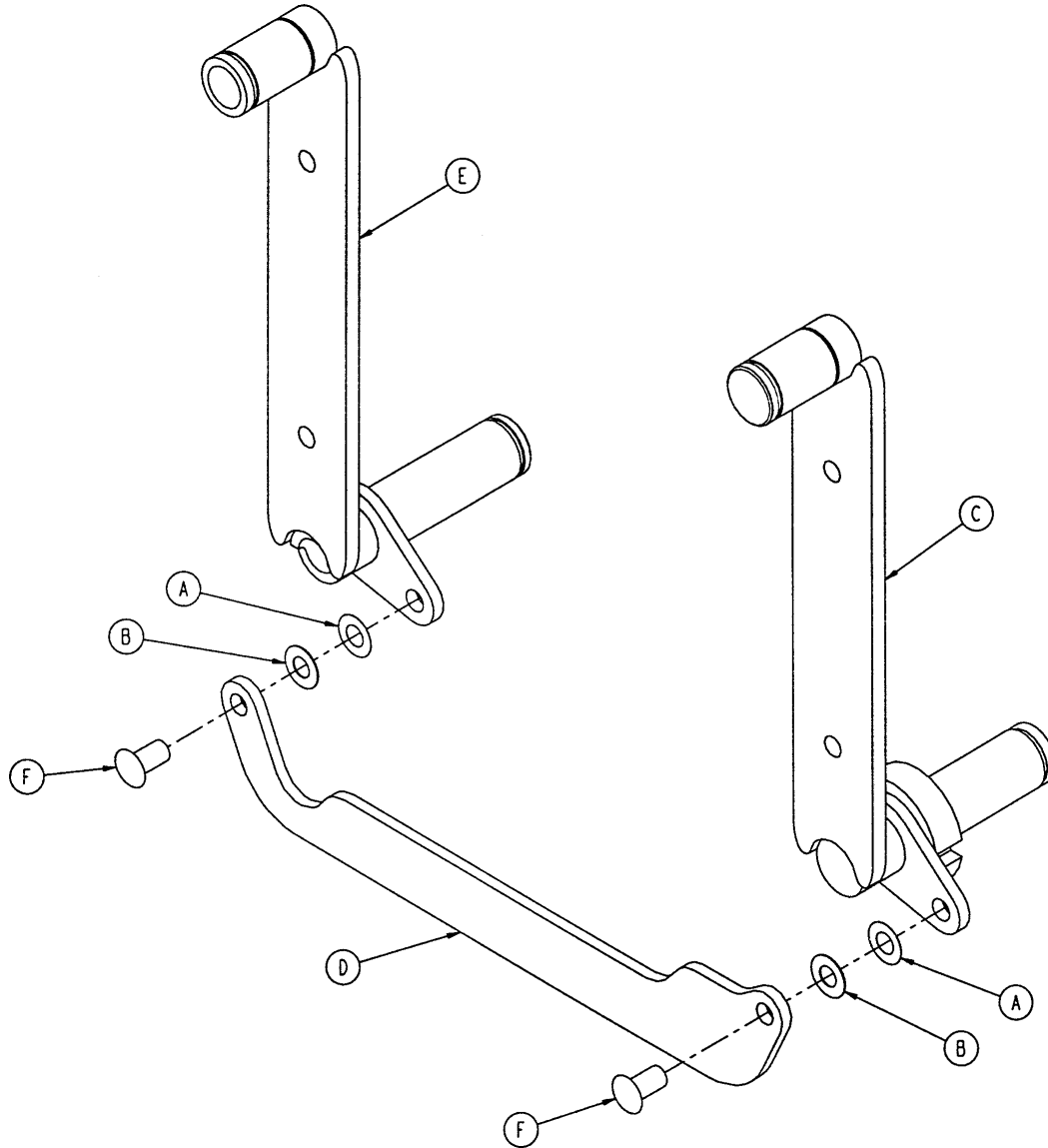


Item	Part No.	Part Name	Qty.
A	11-377	Nylon Washer	2
B	11-403	Shim Washer	2
C	2035-400-427	Arm Wldmt., Right, Foot, Foot	1
D	3001-400-11	Head End Timing Link	1
E	3001-400-128	Arm Wldmt., Left, Foot, Head	1
F	3001-400-501	Siderail Linkage Rivet	2

2035-400-328 Foot End Siderail Timing Link Ass'y, Lt.

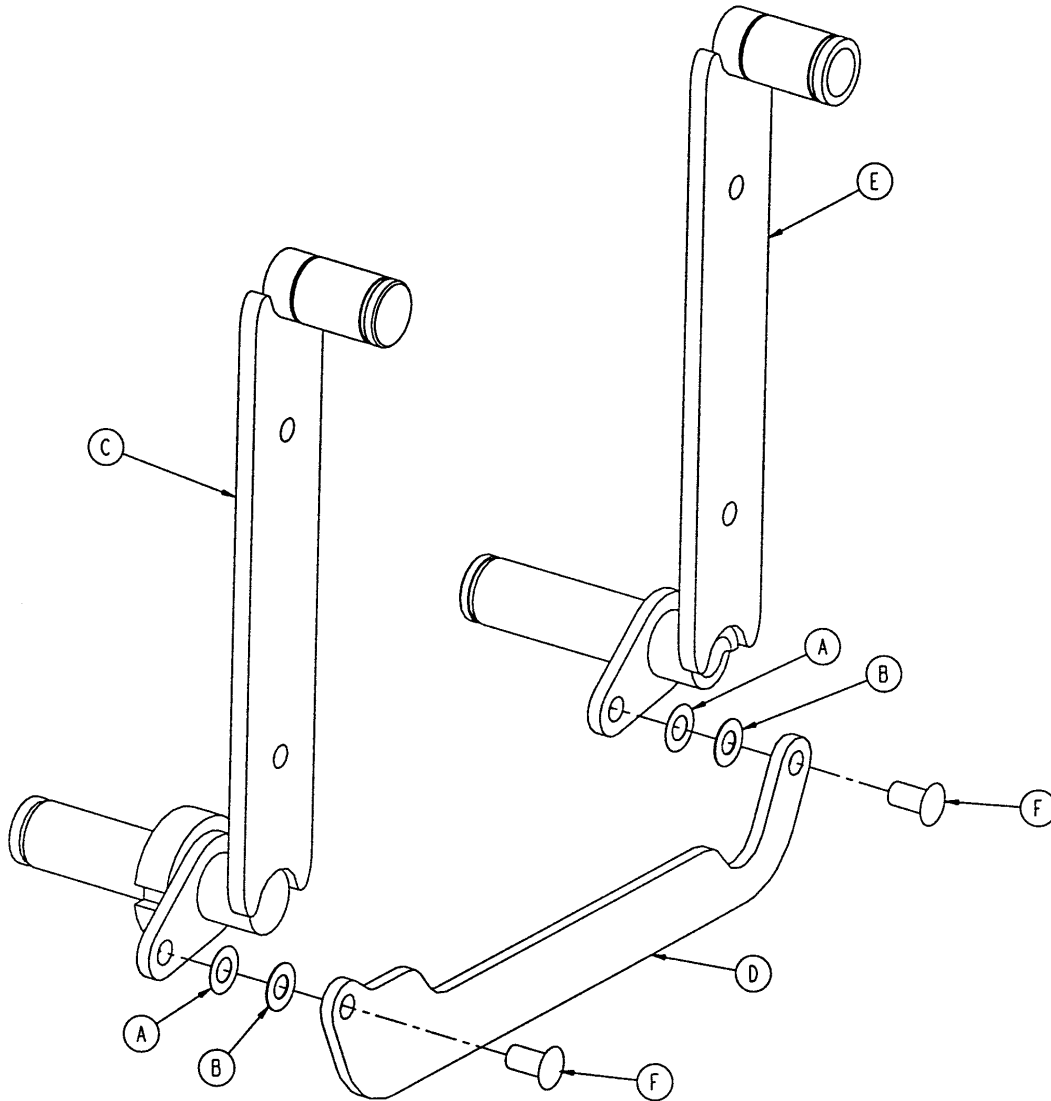

Item	Part No.	Part Name	Qty.
A	11-377	Nylon Washer	2
B	11-403	Shim Washer	2
C	2035-400-327	Arm Wldmt., Left, Foot, Foot	1
D	3001-400-11	Head End Timing Link	1
E	3001-400-228	Arm Wldmt., Right, Foot, Head	1
F	3001-400-501	Siderail Linkage Rivet	2


2035–130–10 Head Board Assembly

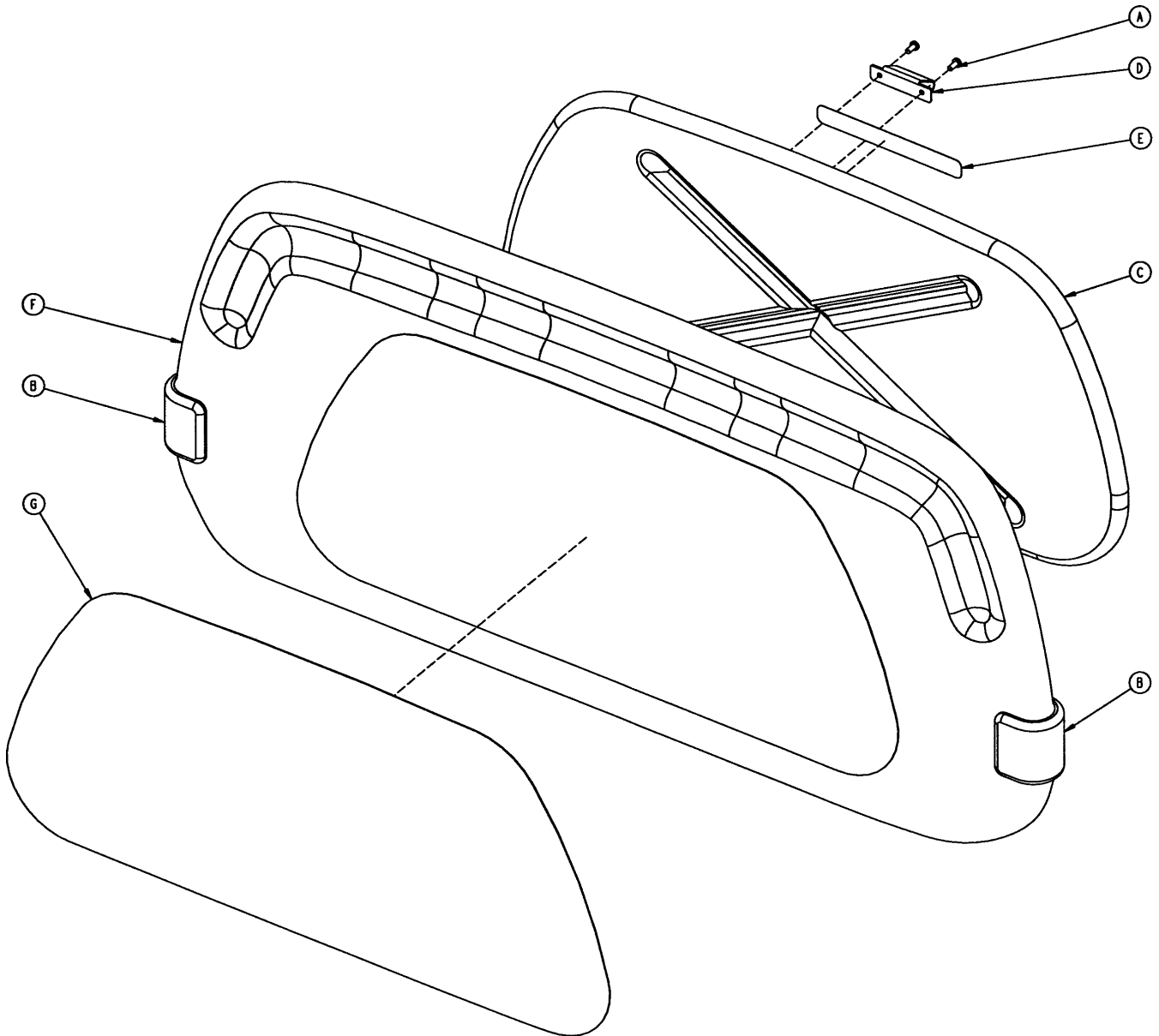

Item	Part No.	Part Name	Qty.
A	23–88	Pan Hd. Screw	2
B	2035–500–7	Dark Blue “C” Bumper	2
C	3000–526–1	CPR Board	1
D	3000–526–2	CPR Board Clip	1
E	3000–526–3	CPR Board Label	1
F	3000–600–10	Head Board Clam Shell Ass’y	1
G	3000–600–56	Beige Head Board Laminate	1
H	72–2–71	“C” Bumper Adhesive	N/A


Foot Board Assembly

part number 2035-000-151


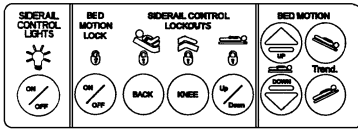


POWER BED MOTION LOCKED

part number 2035-000-155

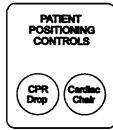

part number 2035-000-153

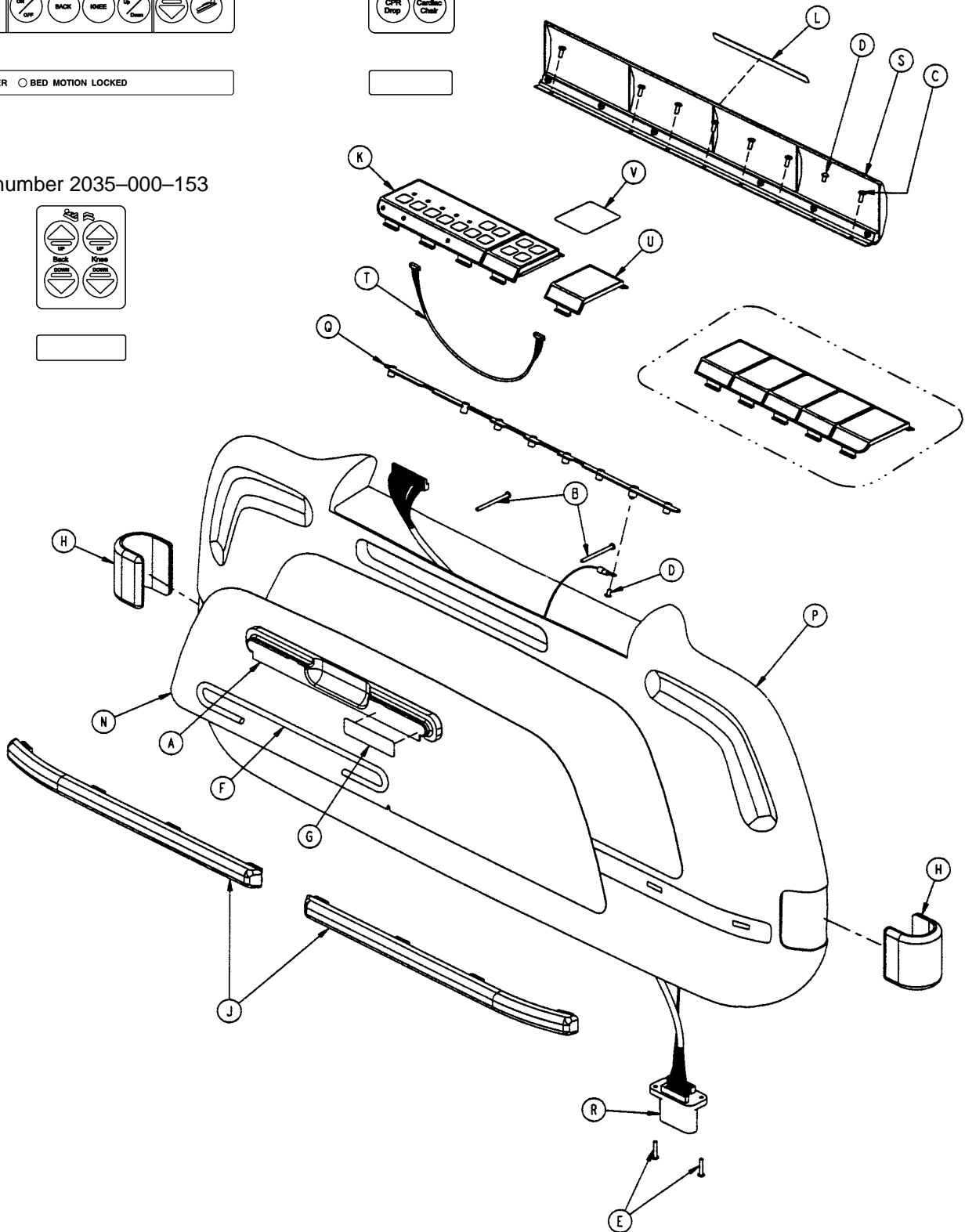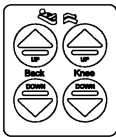


Foot Board Assembly

2035–135–10 Foot Board Assembly – Standard Components

Item	Part No.	Part Name	Qty.
A	3000–525–1	Chart Rack	1
B	23–99	Pan Hd. Screw	2
C	23–103	Pan Hd. Screw	7
D	50–38	Pan Hd. Machine Screw	2
E	50–39	Pan Hd. Machine Screw	2
F	3000–525–2	Chart Rod	1
G	3000–525–4	Pull Handle Label	1
H	2035–500–7	Blue “C” Bumper	2
J	2035–500–8	Bumper Strip	2
K	(page 11–80)	Main Module Assembly	1
L	3000–500–25	Lid Label	1
M	3000–500–29	Caution Fire Hazard Label	1
N	3000–500–56	Beige Laminate	1
P	3001–500–10	Foot Board Clam Shell Ass’y	1
Q	3001–500–64	Hinge Plate	1
R	3001–500–801	Foot Board Cable	1
S	3001–500–1	Lid Assembly	1
T	2025–136–801	E–Drop/Card. Chair Cable	1
U	(page 11–81)	E–Drop/Card. Chair Module	1
V	2035–000–155	E–Drop/Card. Chair Label	1
W	72–2–71	“C” Bumper Adhesive	N/A

Foot Board Assembly


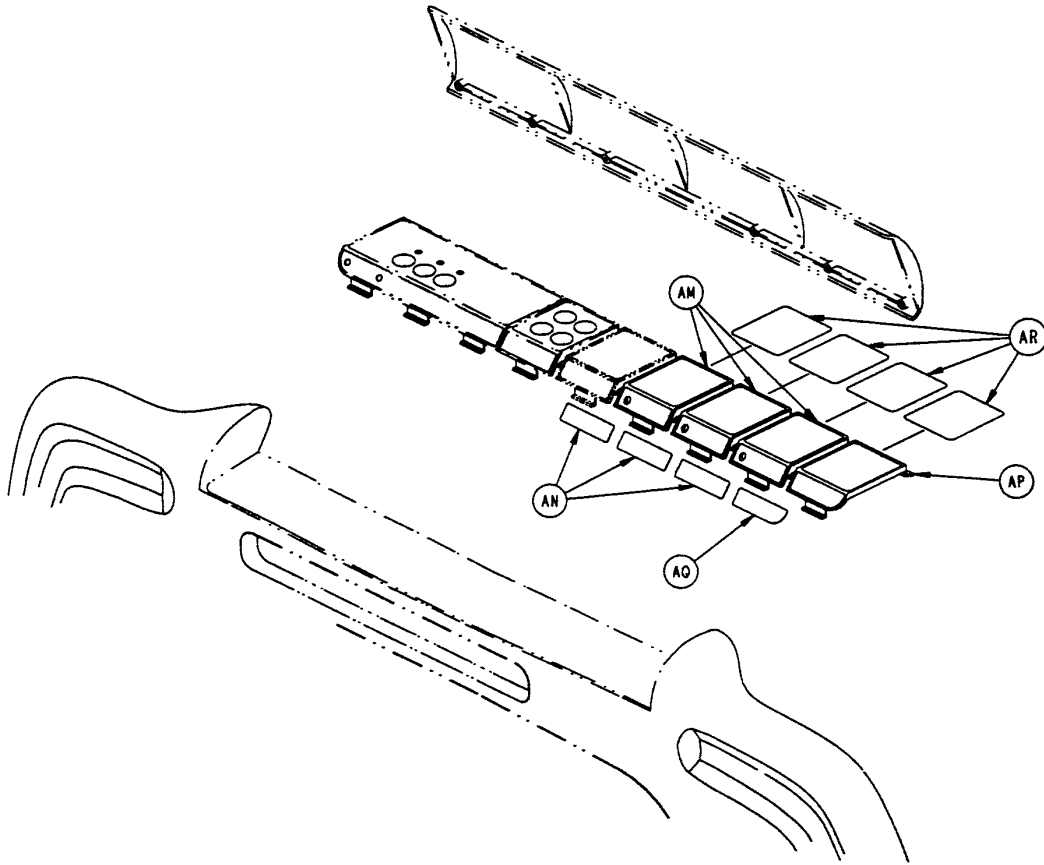


2035-30-101 Foot Board Assembly – No Scale & No Bed Exit

Item	Part No.	Part Name	Qty.
AM	3001-500-3	Blank Module	3
AN	3000-500-26	Blank Module Label	3
AP	3000-500-4	End Module	1
AQ	3000-500-27	Blank End Label	1
AR	2035-500-101	Foot Board Blank Label	4

Foot Board Assembly

part number 2035-000-154

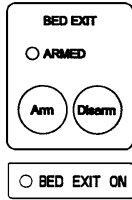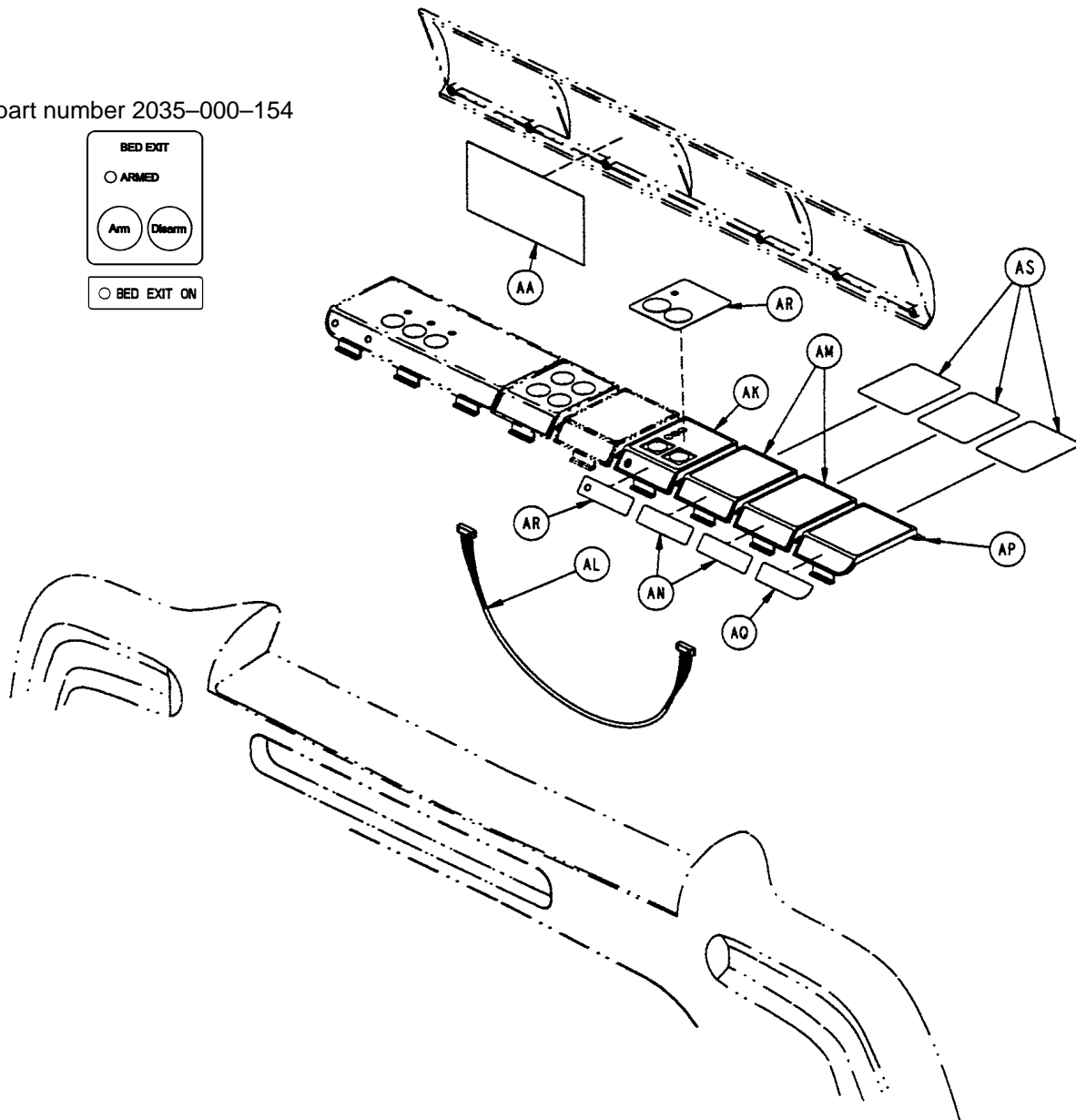


2035-30-176 Foot Board Assembly – Bed Exit Option Only


Item	Part No.	Part Name	Qty.
AA	2025-30-177	Bed Exit Label	1
AD	3001-400-953	Switch Cap	2
AK	(page 11-82)	Bed Exit Module Assembly	1
AL	3001-508-800	Bed Exit Keypad Cable	1
AM	3001-500-3	Blank Module	2
AN	3000-500-26	Blank Module Label	2
AP	3000-500-4	End Module	1
AQ	3000-500-27	Blank End Label	1
AR	2035-000-154	Bed Exit Label	1
AS	2035-500-101	Foot Board Blank Label	3

Foot Board Assembly

part number 2035-000-152

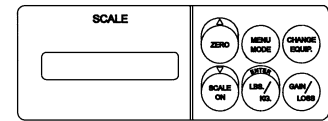


stryker®

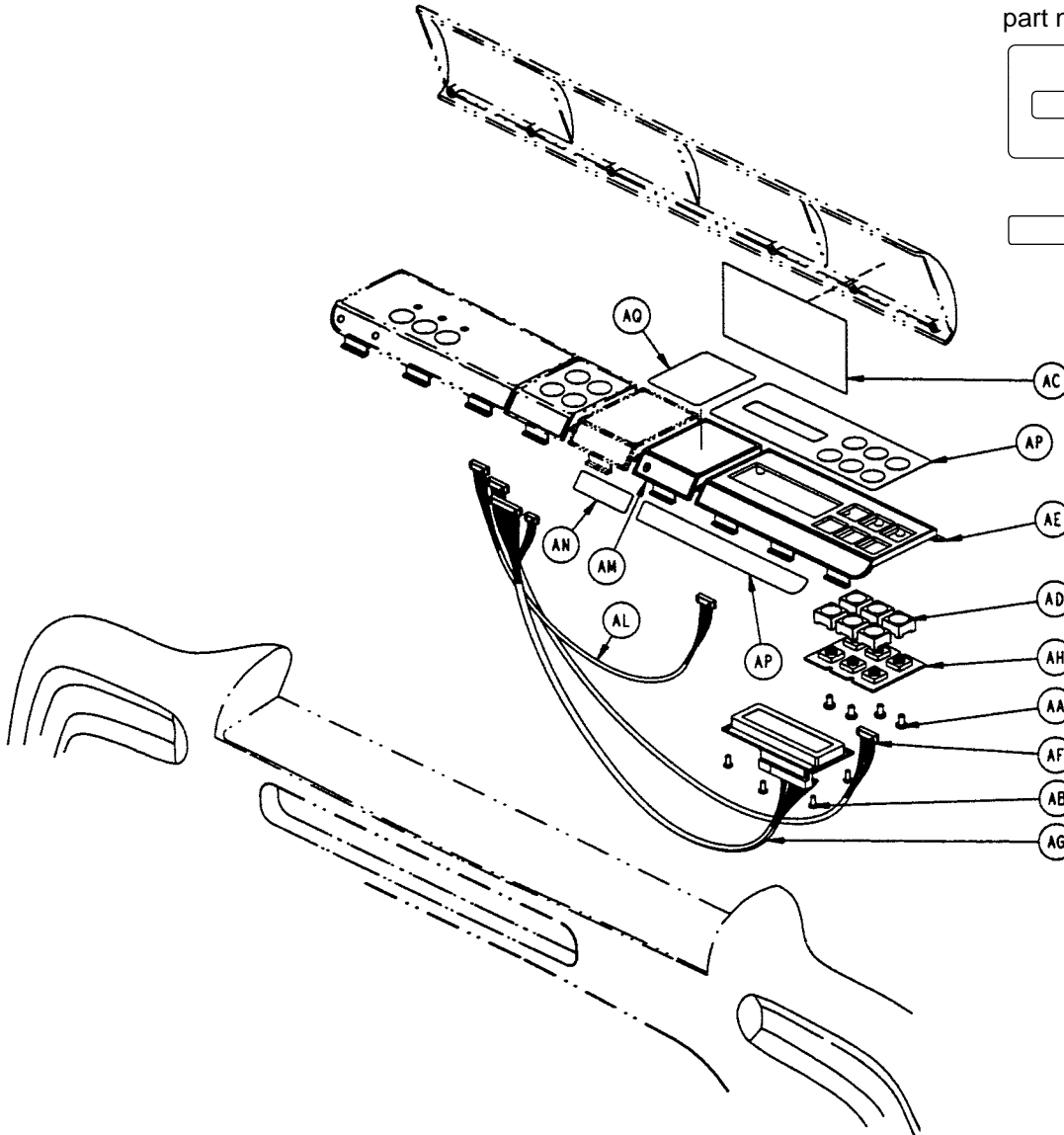

2035-30-126 Foot Board Assembly – Scale Option Only

Item	Part No.	Part Name	Qty.
AA	23-87	Pan Hd. Tapping Screw	4
AB	23-91	Pan Hd. Tapping Screw	4
AC	2025-30-127	Scale Lid Label	1
AD	3001-400-953	Switch Cap	6
AE	3001-507-1	Scale Module	1
AF	3001-507-800	Scale Keypad Cable	1
AG	3001-507-900	Scale LCD Display Cable	1
AH	3001-507-910	Scale Keypad PCB	1
AM	3001-500-3	Blank Panel	1
AN	3000-500-26	Blank Module Label	1
AP	2035-000-152	Scale Label	1
AQ	2035-500-101	Foot Board Blank Label	1

Foot Board Assembly

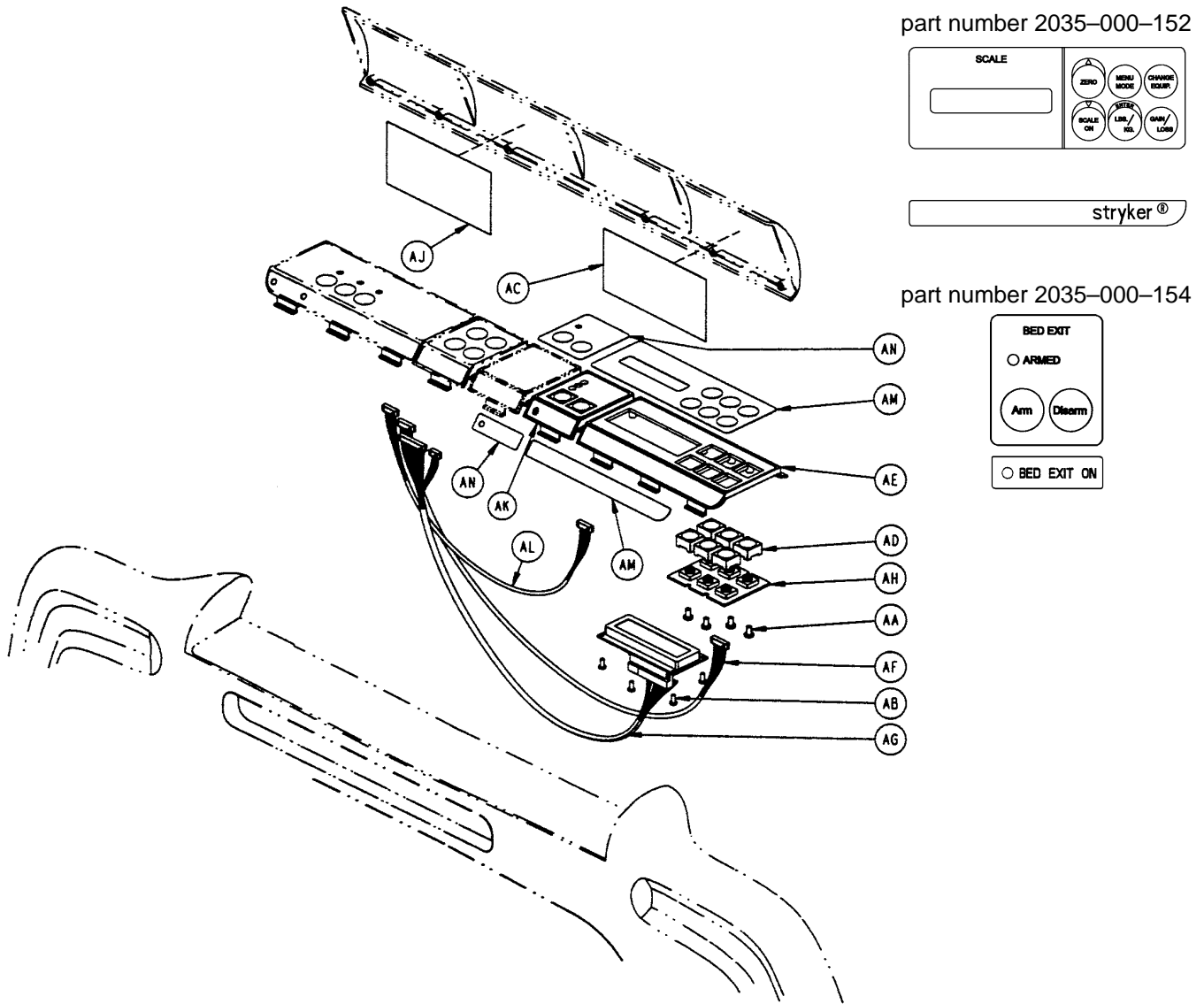


2035-30-151 Foot Board Assembly – Scale and Center of Gravity Bed Exit Options

Item	Part No.	Part Name	Qty.
AA	23-87	Pan Hd. Tapping Screw	4
AB	23-91	Pan Hd. Tapping Screw	4
AC	2025-30-127	Scale Lid Label	1
AD	3001-400-953	Switch Cap	6
AE	3001-507-1	Scale Module	1
AF	3001-507-800	Scale Keypad Cable	1
AG	3001-507-900	Scale LCD Display Cable	1
AH	3001-507-910	Scale Keypad PCB	1
AJ	2025-30-177	Bed Exit Lid Label	1
AK	(page 11-82)	Bed Exit Module Assembly	1
AL	3001-508-800	Bed Exit Keypad Cable	1
AM	2035-000-152	Scale Label	1
AN	2035-000-154	Bed Exit Label	1



2035–235–20 Foot Board Main Module Assembly

part number 2035–000–151

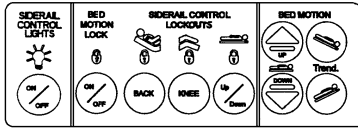


POWER BED MOTION LOCKED

part number 2035–000–153

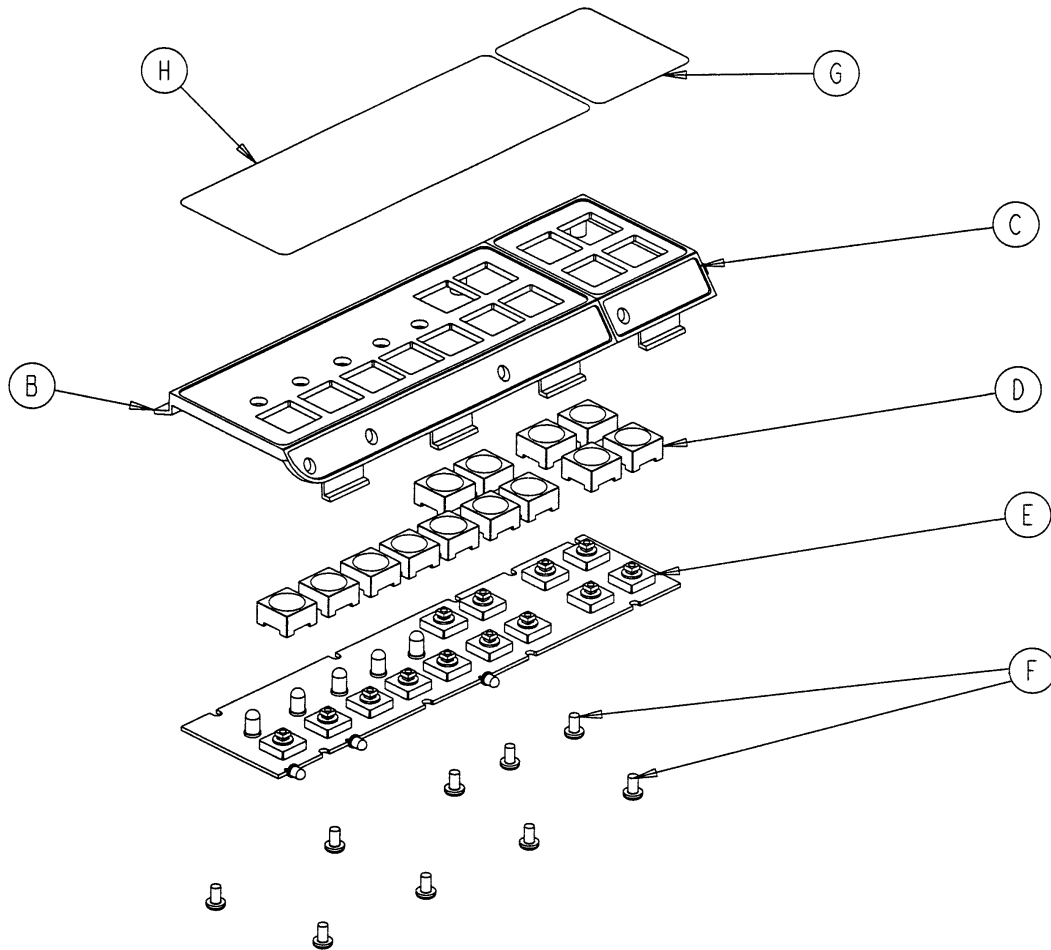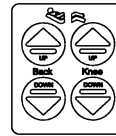


Item	Part No.	Part Name	Qty.
B	3000–500–2	Foot Board Standard Module	1
C	3000–501–1	Gatch/Fowler Module	1
D	3001–400–953	Switch Cap	13
E	3001–500–930	Main Foot Board PCB	1
F	23–87	Pan Hd. Tapping Screw	9
G	2035–000–153	Gatch/Fowler Label	1
H	2035–000–151	Foot Board Std. Module Label	1

2025–136–21 Foot Board CPR Drop/Cardiac Chair Module Ass’y

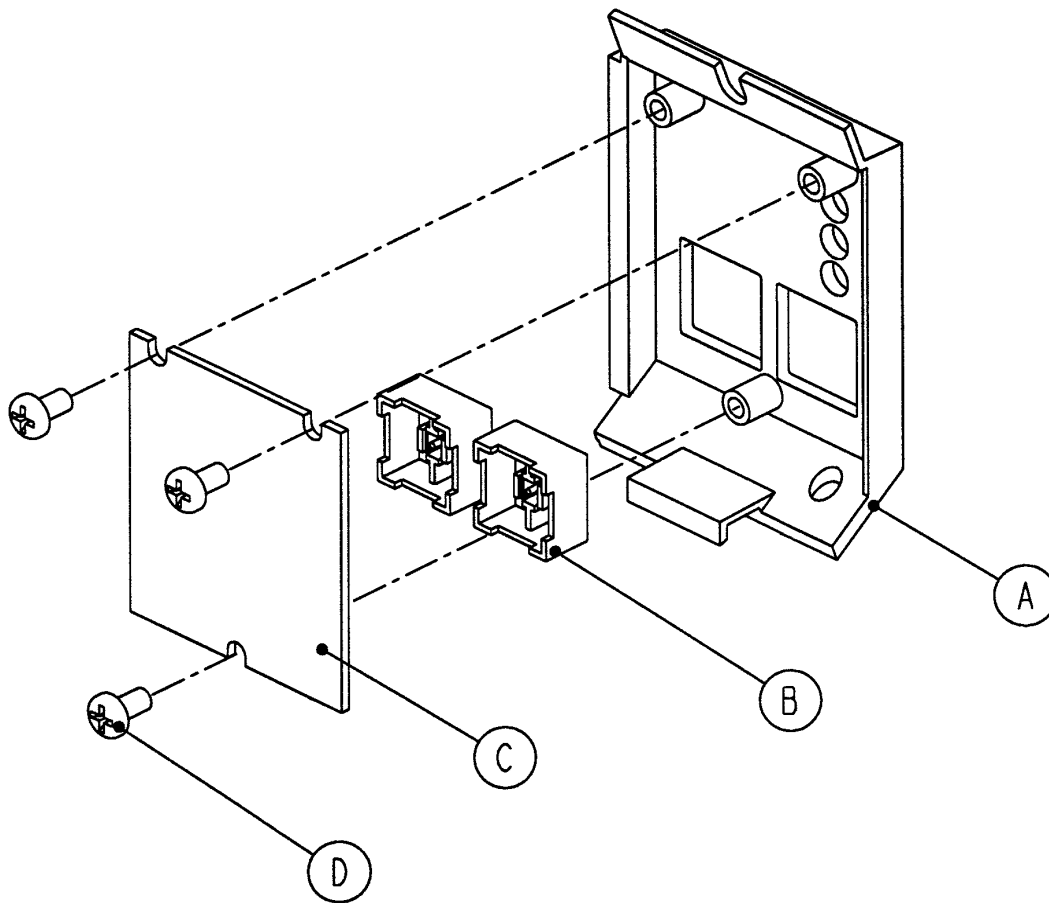


Item	Part No.	Part Name	Qty.
A	3000–508–1	Bed Exit Module Panel	1
B	3001–400–953	Switch Cap	2
C	2025–136–900	CPR Drop/Card. Ch. Keypad	1
D	23–87	Hi–Low Tapping Screw	3

2025–136–22 Foot Board Bed Exit Module Assembly

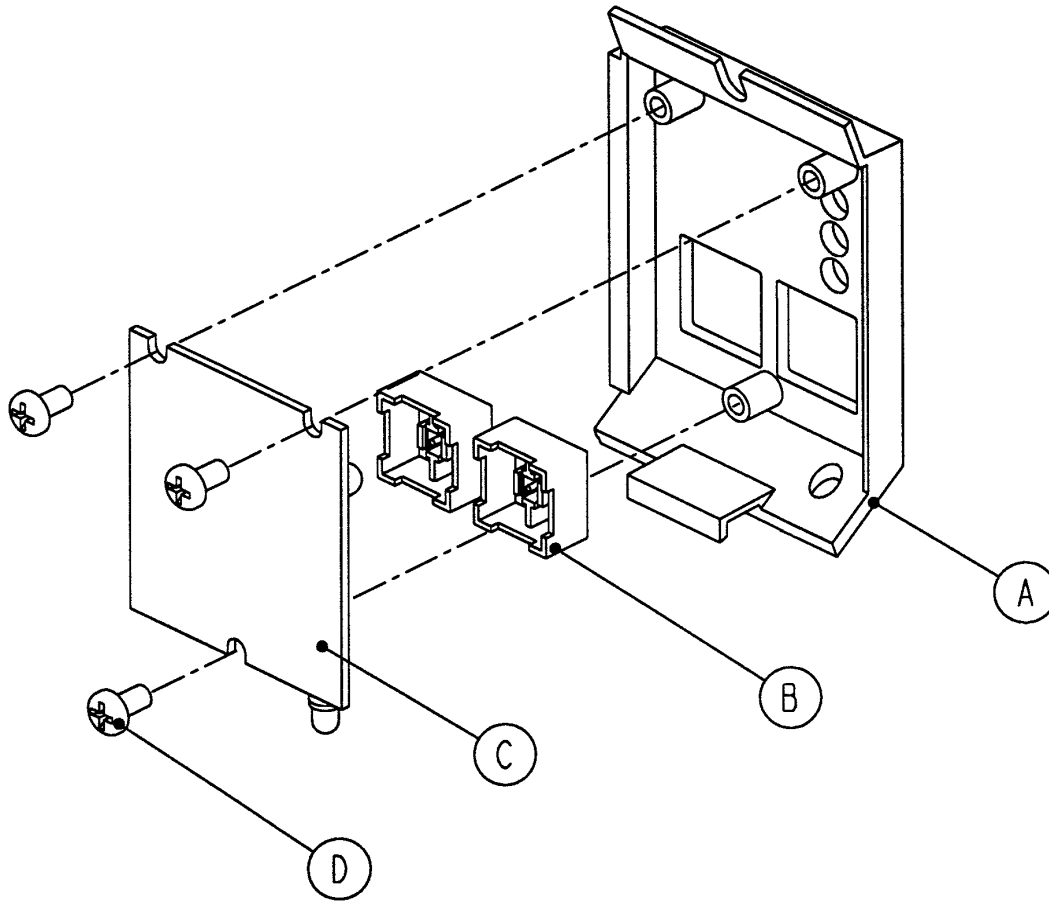

Item	Part No.	Part Name	Qty.
A	3000–508–1	Bed Exit Module Panel	1
B	3001–400–953	Switch Cap	2
C	3001–508–910	Bed Exit Keypad Ass'y	1
D	23–87	Hi–Low Tapping Screw	3


Optional Pendant Assembly

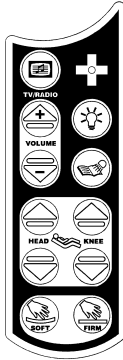


**3001-315-11 Combination Pendant
Motion/Comm./DMS**




**3001-315-16 Combination Pendant
Communication Only**

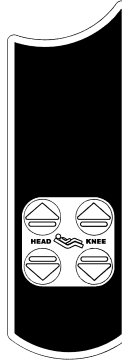

**3001-315-14 Combination Pendant
Motion Only**

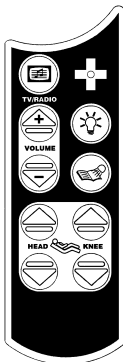


**3001-315-12 Combination Pendant
Motion/Communication**

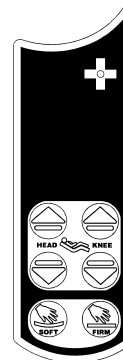


**3001-315-17 Combination Pendant
Motion/DMS/NC**



**3001-315-15 Combination Pendant
Communication/DMS**

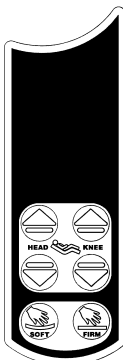


**3001-315-13 Combination Pendant
Motion/DMS**

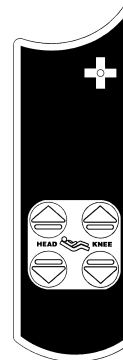


**3001-315-18 Combination Pendant
Motion/NurseCall**

3000–300–80 Removable I.V. Pole Assembly

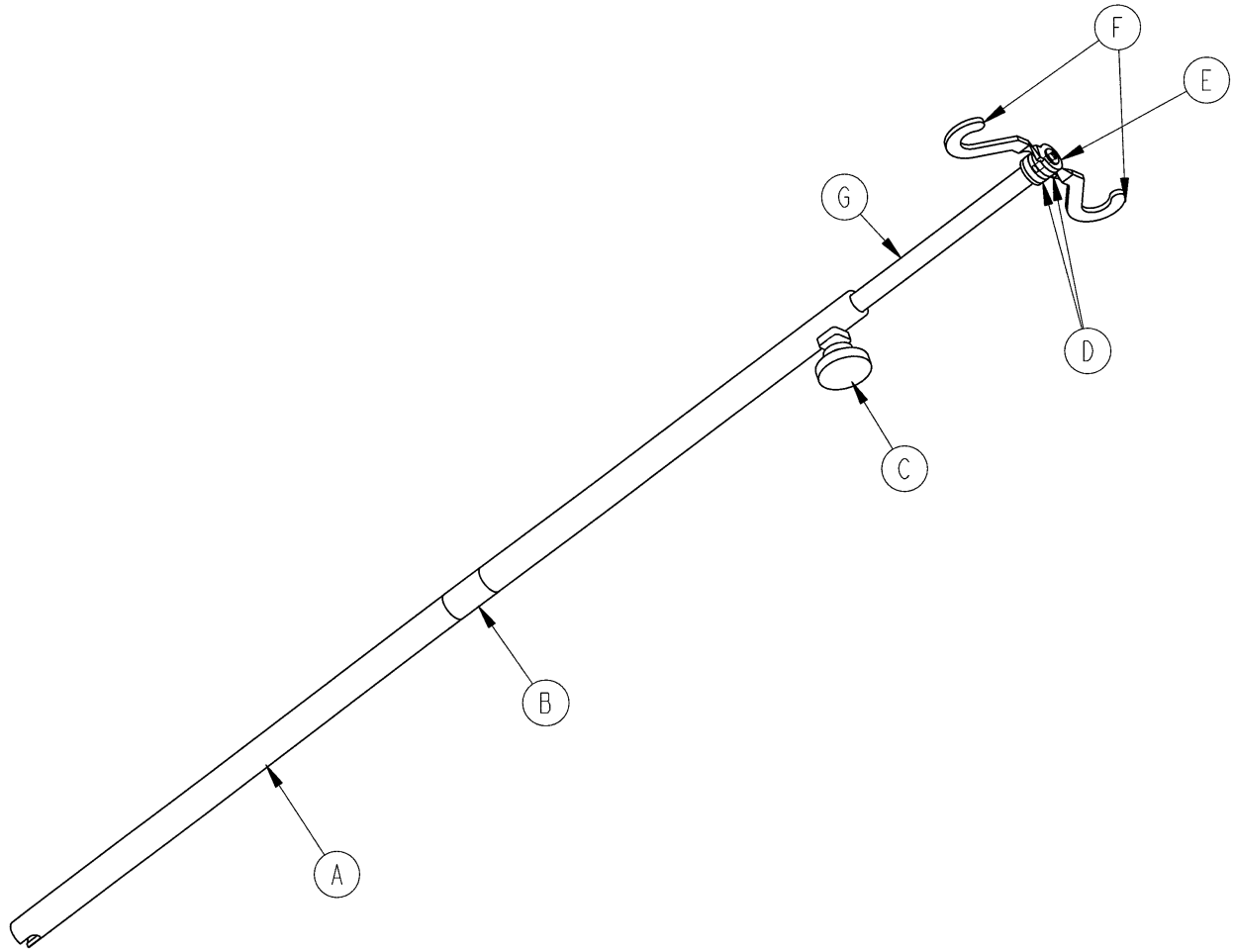


Item	Part No.	Part Name	Qty.
A	3000–300–81	Outer Tube	1
B	3000–300–89	Label	1
C	24–50	Fluted Knob	1
D	52–17	Spacer	2
E	7–40	Phillips Truss Hd. Screw	1
F	1010–59–16	I.V. Hook	2
G	3000–300–85	Inner Tube Assembly	1

2035-111 Optional 2-Stage I.V. Mounting Assembly, Foot End

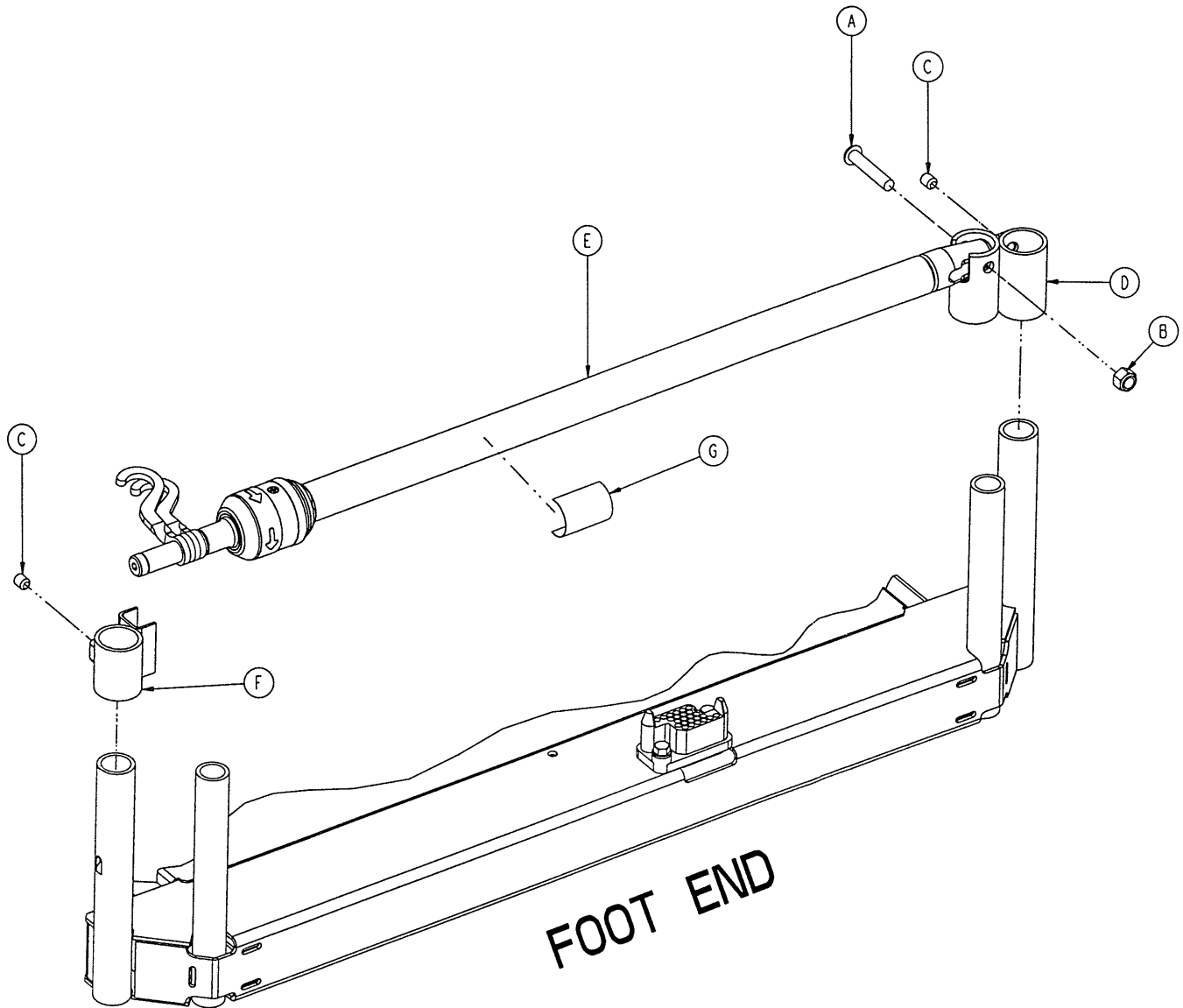


Item	Part No.	Part Name	Qty.
A	4-199	But. Hd. Cap Screw	1
B	16-11	Flexlock Nut	1
C	21-140	Set Screw	2
D	2035-111-1	I.V. Receptacle, Foot, Left	1
E	(page 11-88)	I.V. Pole Assembly, Left	1
F	3000-312-35	I.V. Cradle	1
G	2035-112-110	Specification Label	1

2035–112 Optional 2–Stage I.V. Mounting Assembly, Head End

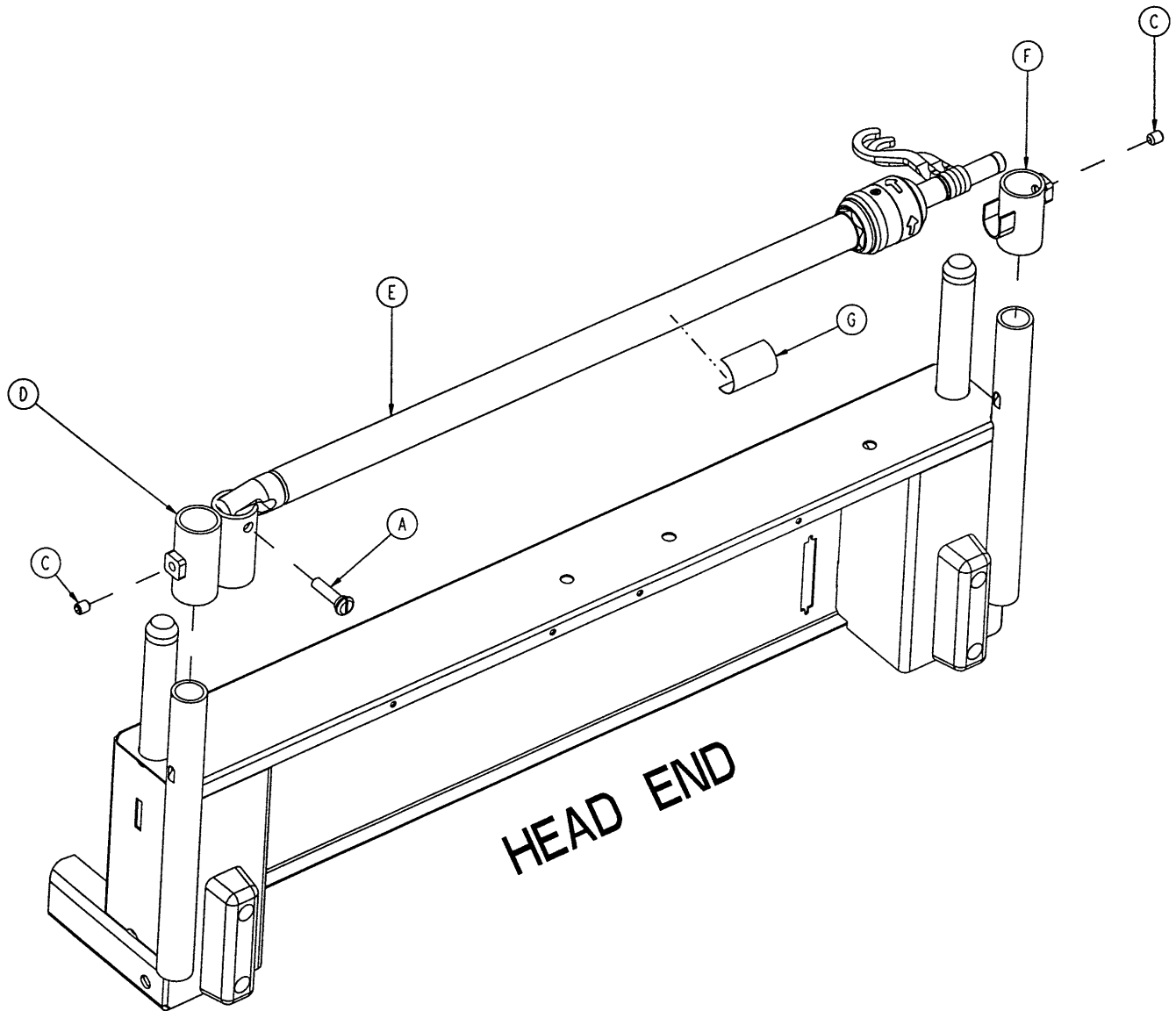


Item	Part No.	Part Name	Qty.
A	1015–24–35	Retaining Pin	1
C	21–140	Set Screw	2
D	2035–112–1	I.V. Receptacle, Head, Left	1
E	(page page 11–88)	I.V. Pole Assembly, Left	1
F	3000–311–16	I.V. Rest	1
G	2035–112–110	Specification Label	1

2035-113 Optional 2-Stage I.V. Mounting Ass'y, Dual Head End

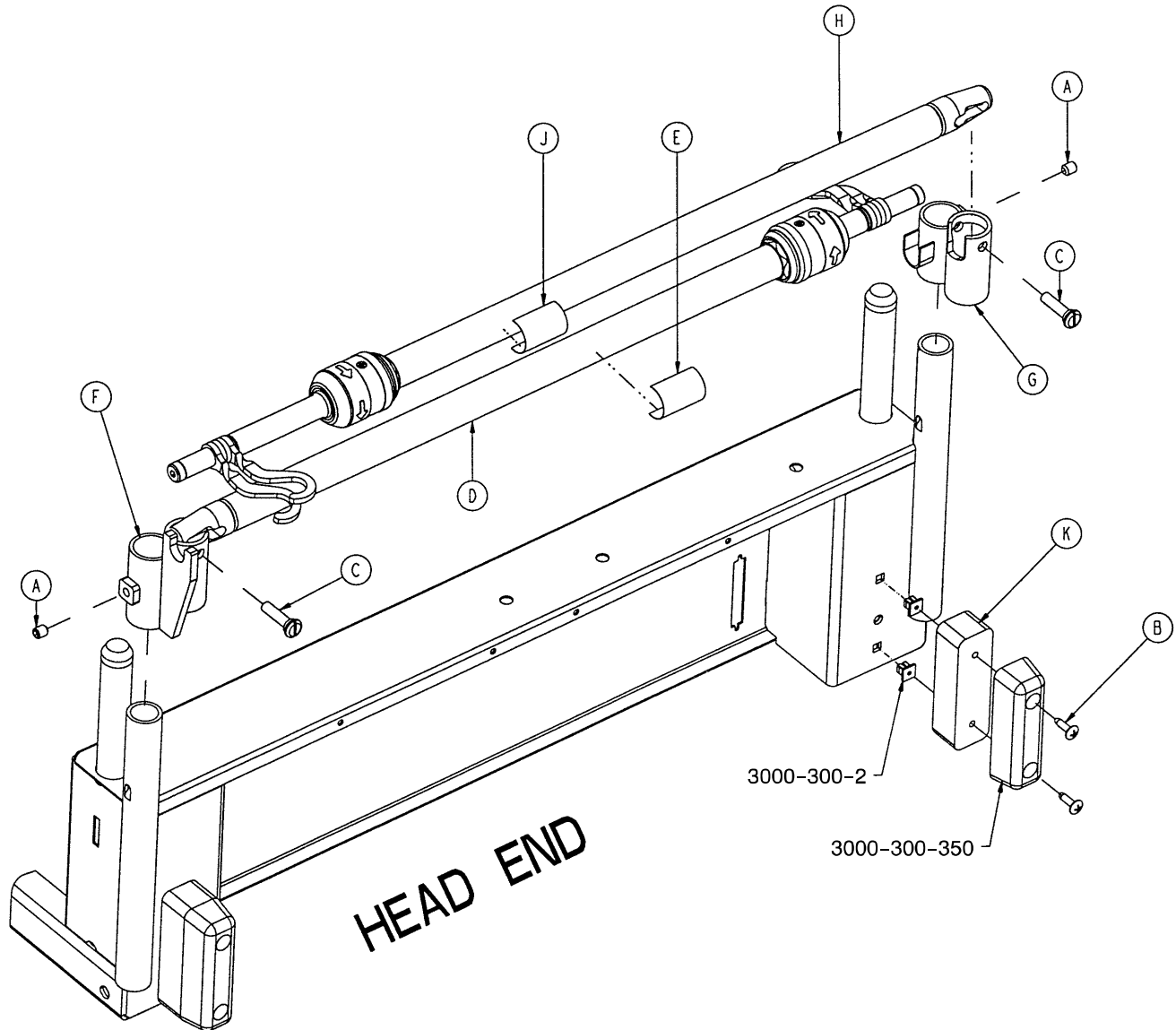

Item	Part No.	Part Name	Qty.
A	21-140	Set Screw	2
B	23-277	Truss Hd. Screw	4
C	1015-24-35	Retaining Pin	2
D	(page 11-88)	I.V. Pole Assembly, Left	1
E	2035-112-110	Specification Label	1
F	2035-113-1	I.V. Receptacle, Dual Head, Lt.	1
G	2035-113-2	I.V. Receptacle, Dual Head, Rt.	1
H	(page 11-88)	I.V. Pole Assembly, Right	1
J	2035-113-111	Specification Label	1
K	2035-113-6	Head End Bumper Spacer	2

2035-112-10 & 2035-113-11 Optional 2-Stage I.V. Ass’y, Hd. & Ft.

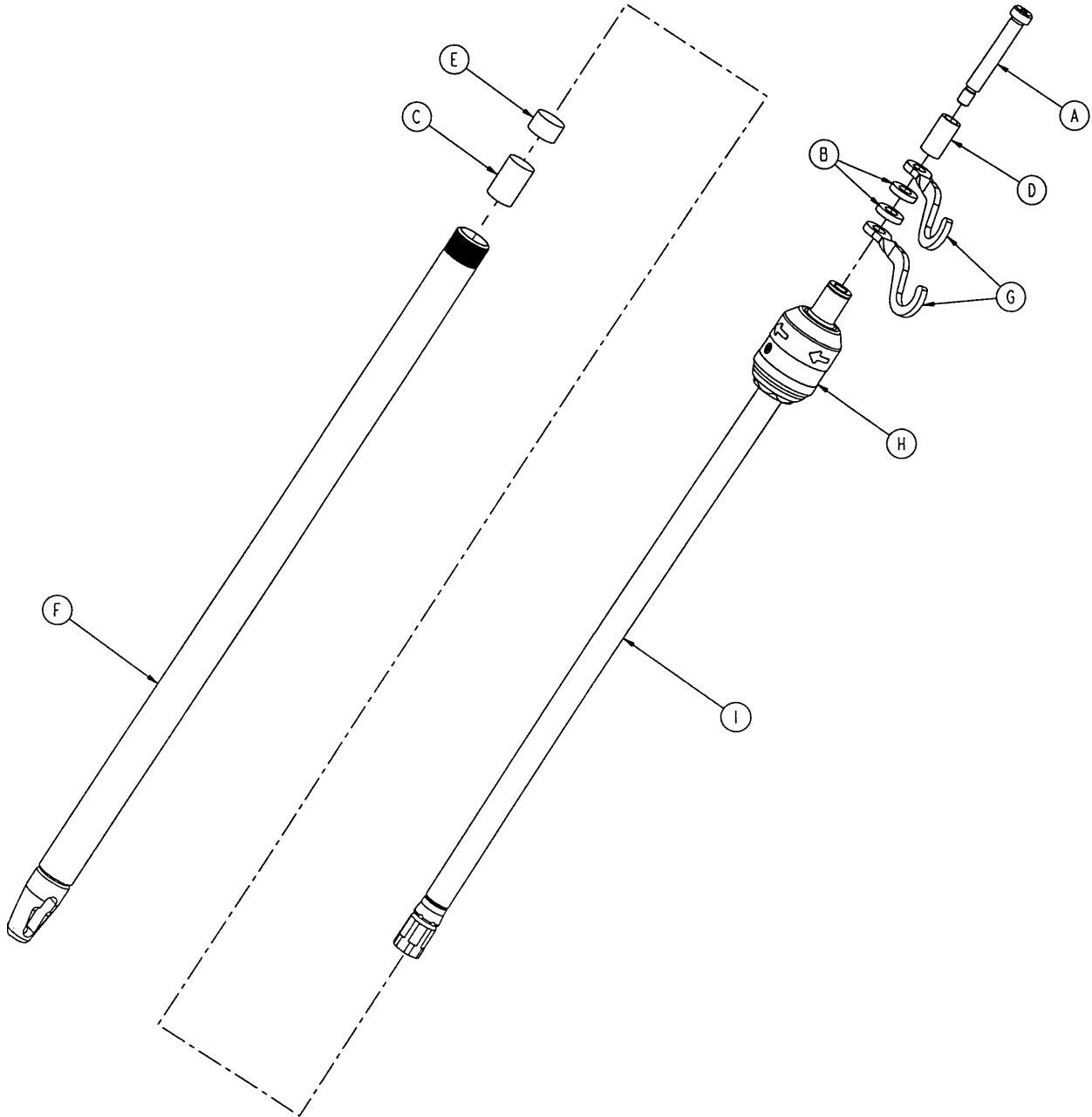


2035-112-10 Head End, Left

2035-113-11 Foot End, Right

Item	Part No.	Part Name	Qty.	Item	Part No.	Part Name	Qty.
A	8-31	Soc. Hd. Shoulder Screw	1	A	8-31	Soc. Hd. Shoulder Screw	1
B	52-17	Washer	2	B	52-17	Washer	2
C	52-310	Spacer	1	C	52-311	Spacer	1
D	926-400-162	Spacer	1	D	926-400-162	Spacer	1
E	1001-259-13	Dampener	1	E	1001-259-13	Dampener	1
F	1001-259-32	Base Tube Weldment	1	F	1001-259-32	Base Tube Weldment	1
G	1010-259-16	I.V. Hook	2	G	1010-259-16	I.V. Hook	2
H	(page 11-89)	I.V. Pole Latch	1	H	(page 11-89)	I.V. Pole Latch	1
I	1211-110-29	2nd Stage Assembly	1	I	1211-110-29	2nd Stage Assembly	1


1211-210-26 Optional I.V. Pole Latch Assembly

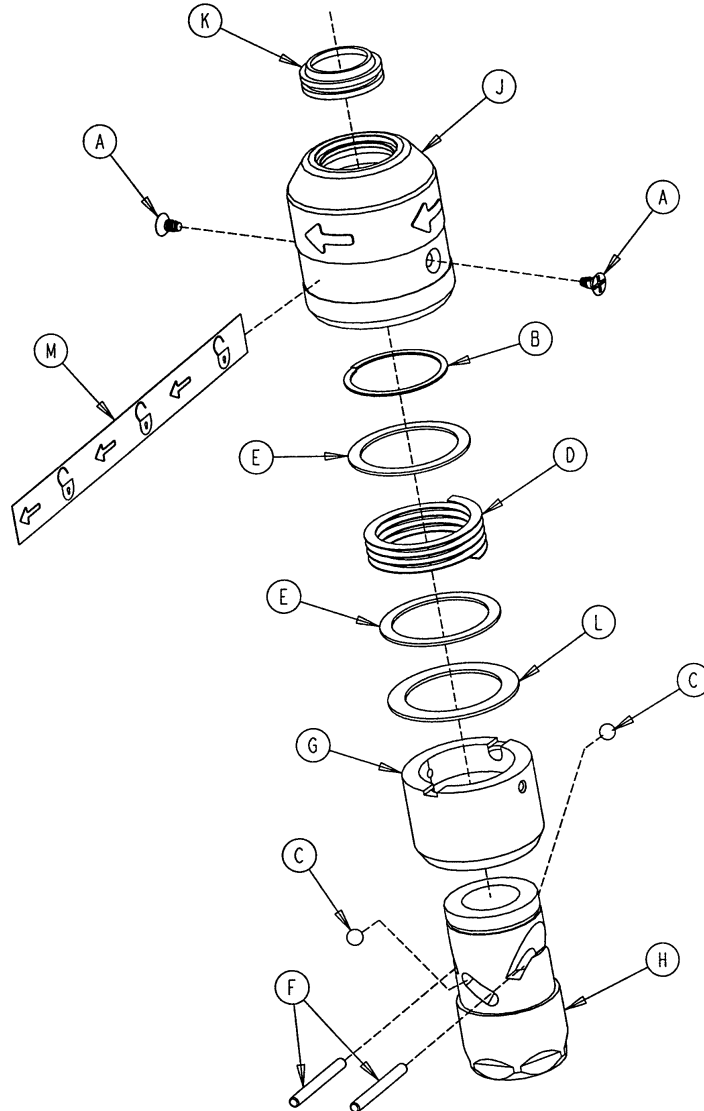

Item	Part No.	Part Name	Qty.
A	1211-110-36	Self-Tapping Screw	2
B	28-167	External Retaining Ring	1
C	31-4	Steel Ball	2
D	38-392	Crest-to Crest Spring	1
E	1211-110-20	Washer	2
F	1211-110-21	Locking Pin	2
G	1211-110-22	Guide	1
H	1211-110-23	I. D. Housing	1
J	2035-110-24	O. D. Housing	1
K	1211-110-18	I.V. Latch Seal	1
L	1211-110-35	Washer	1
M	1211-91-34	Release Label	1


2035-140 Optional Fowler X-Ray Cassette Holder Assembly

Assembly part number
2035-140-10
(reference only)

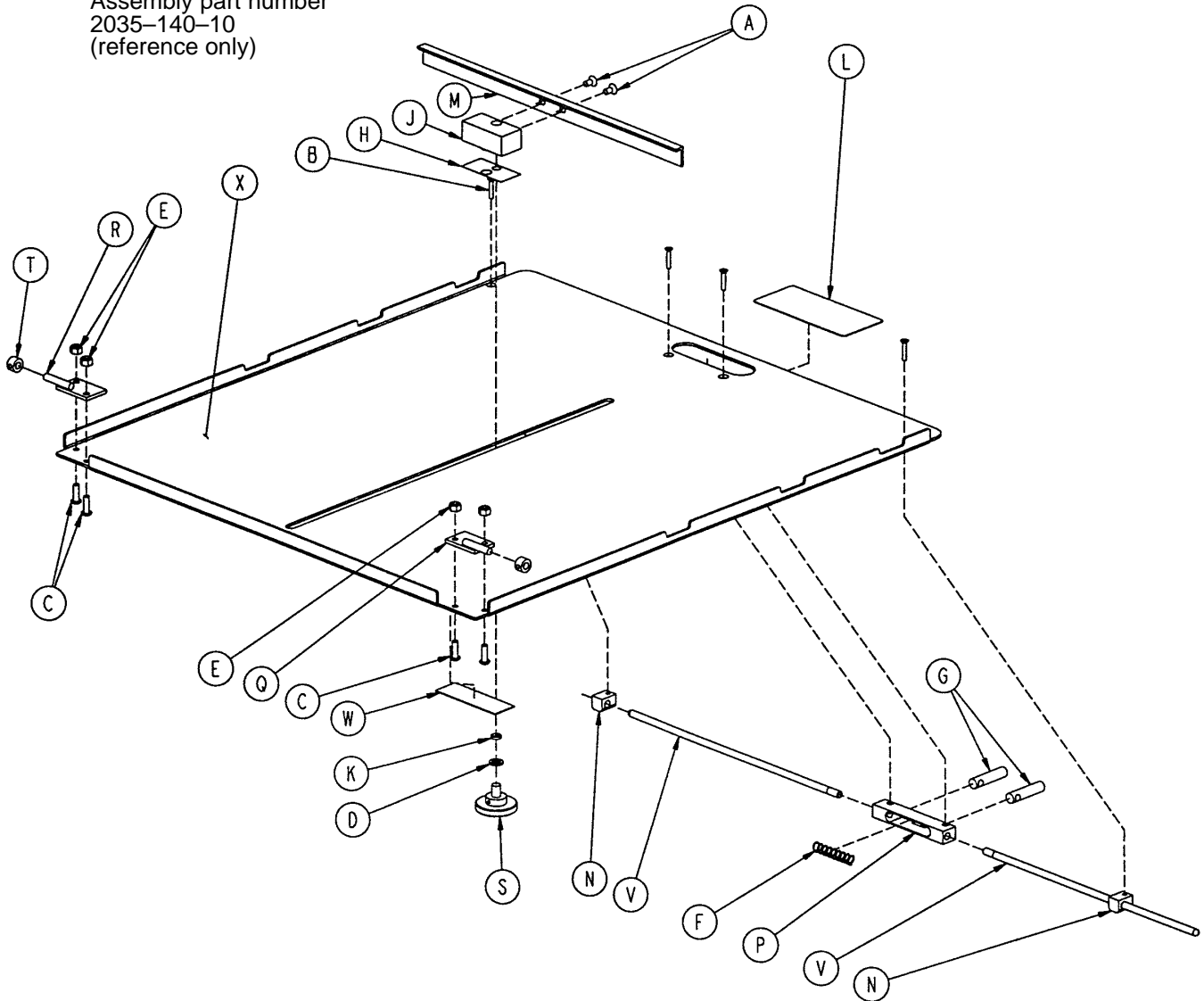


Item	Part No.	Part Name	Qty.	Item	Part No.	Part Name	Qty.
A	1-20	Flat C'sunk Hd. Mach Scr.	2	M	1010-23-28	Tray Angle	1
B	1-22	Flat C'sunk Hd. Mach Scr.	4	N	1010-23-37	Cassette Rod Guide	2
C	4-149	H. Soc. But. Hd. Cap Scr.	4	P	1020-23-16	Cassette Post Housing	1
D	14-3	Washer	1	Q	1020-23-19	Tray Hinge Wldmt., Rt.	1
E	16-3	Hex Nut	4	R	1020-23-20	Tray Hinge Wldmt., Lt.	1
F	38-122	Spring	1	S	1020-23-21	Knob	1
G	926-23-64	Tray Post	2	T	42-13	Collar w/Set Screw	2
H	926-23-69	Cassette Washer	1	V	2025-140-2	Cassette Actuating Rod	2
J	926-23-70	Cassette Block Subass'y	1	W	2035-140-25	Specification Label	1
K	926-23-71	Cassette Bushing	1	X	2035-140-99	Cassette Tray	1
L	1010-23-19	Instruction Label	1				

2035–19–10 Optional Siderail Transducer Mount Assembly

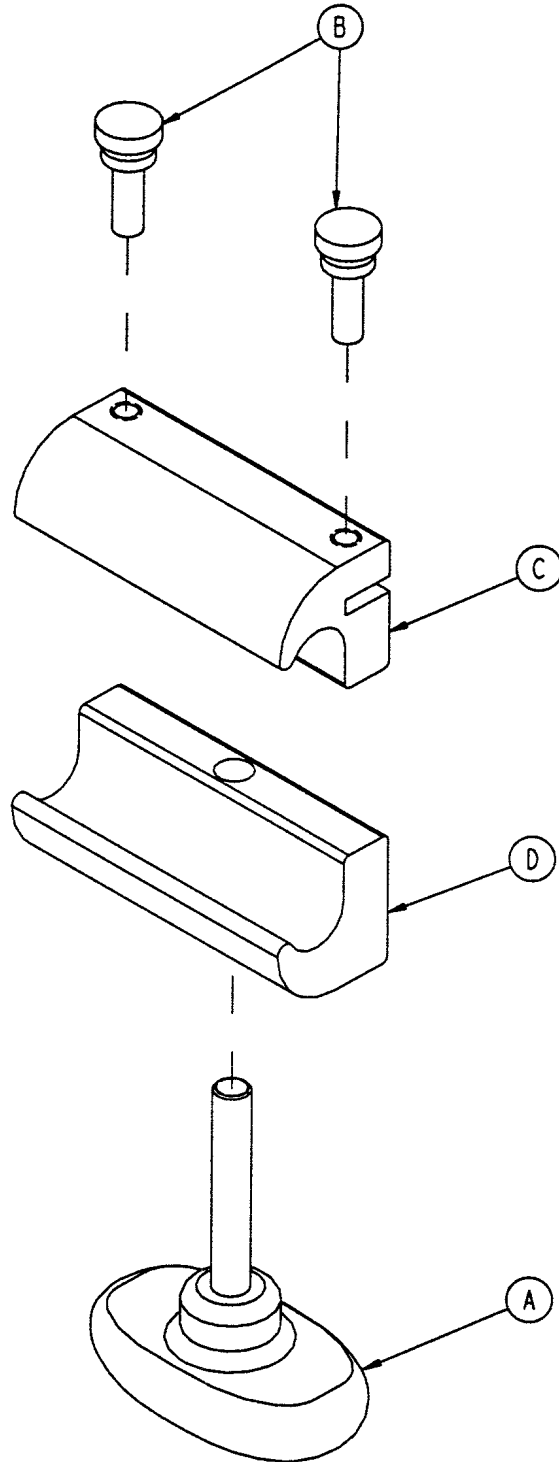


Item	Part No.	Part Name	Qty.
A	24-63	T-Knob	1
B	24-64	Thumb Screw	2
C	2035-19-11	Transducer Mount, Top	1
D	2035-19-12	Transducer Mount, Bottom	1

2035–18–10 Optional I.V. Pole Transducer Mount Assembly

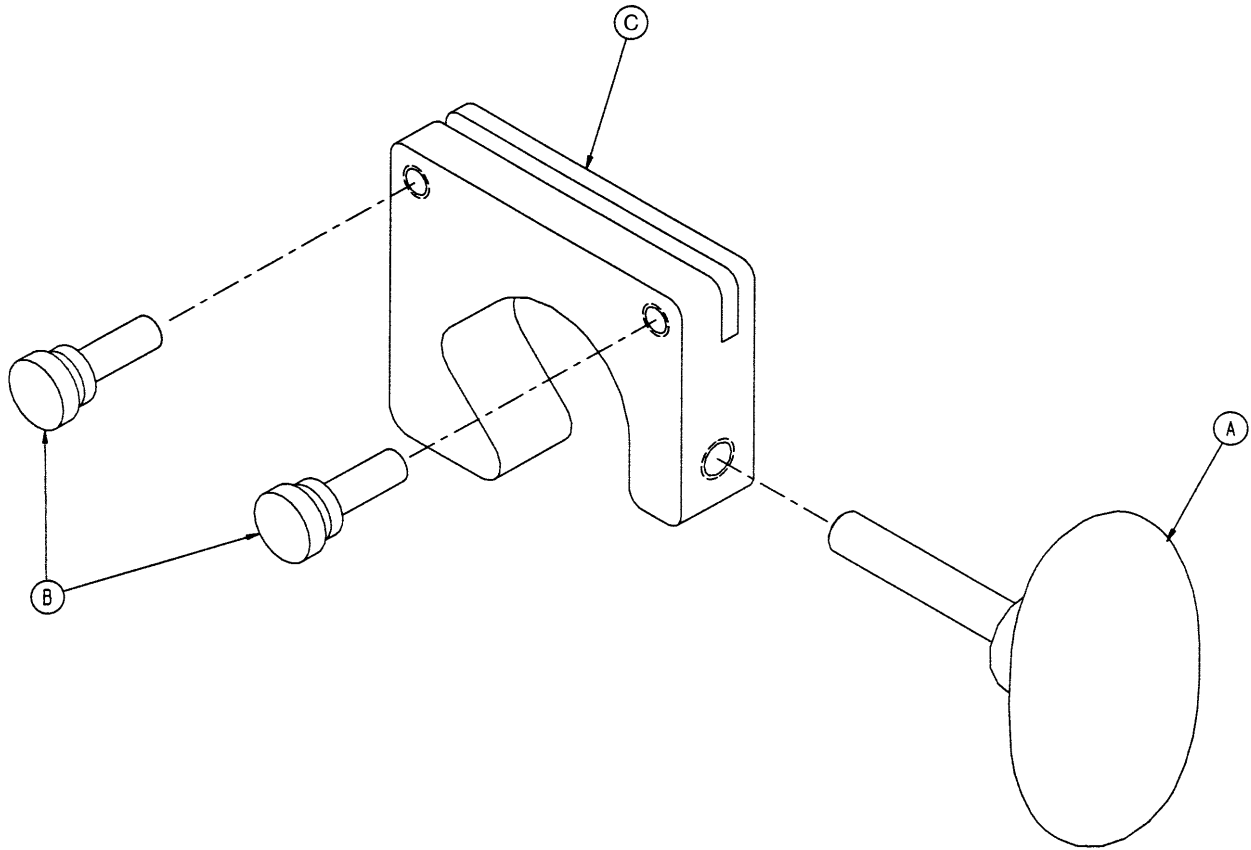


Item	Part No.	Part Name	Qty.
A	24-63	T-Knob	1
B	24-64	Thumb Screw	2
C	2035-18-11	Transducer Mount	1

2025-150-10 Optional Upright O2 Bottle Holder Assembly

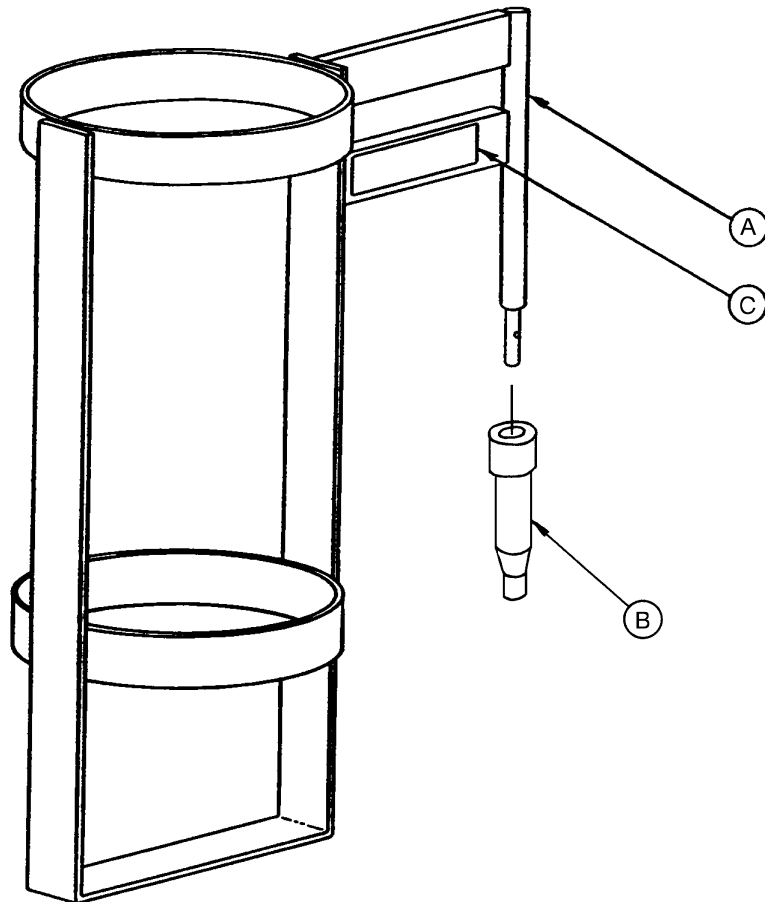


Item	Part No.	Part Name	Qty.
A	1010-30-11	Upright Bottle Holder	1
B	2025-150-1	Bottle Holder Adapter	1
C	2025-150-2	Specification Label	1

2025-120 Optional Defibrillator Tray Assembly

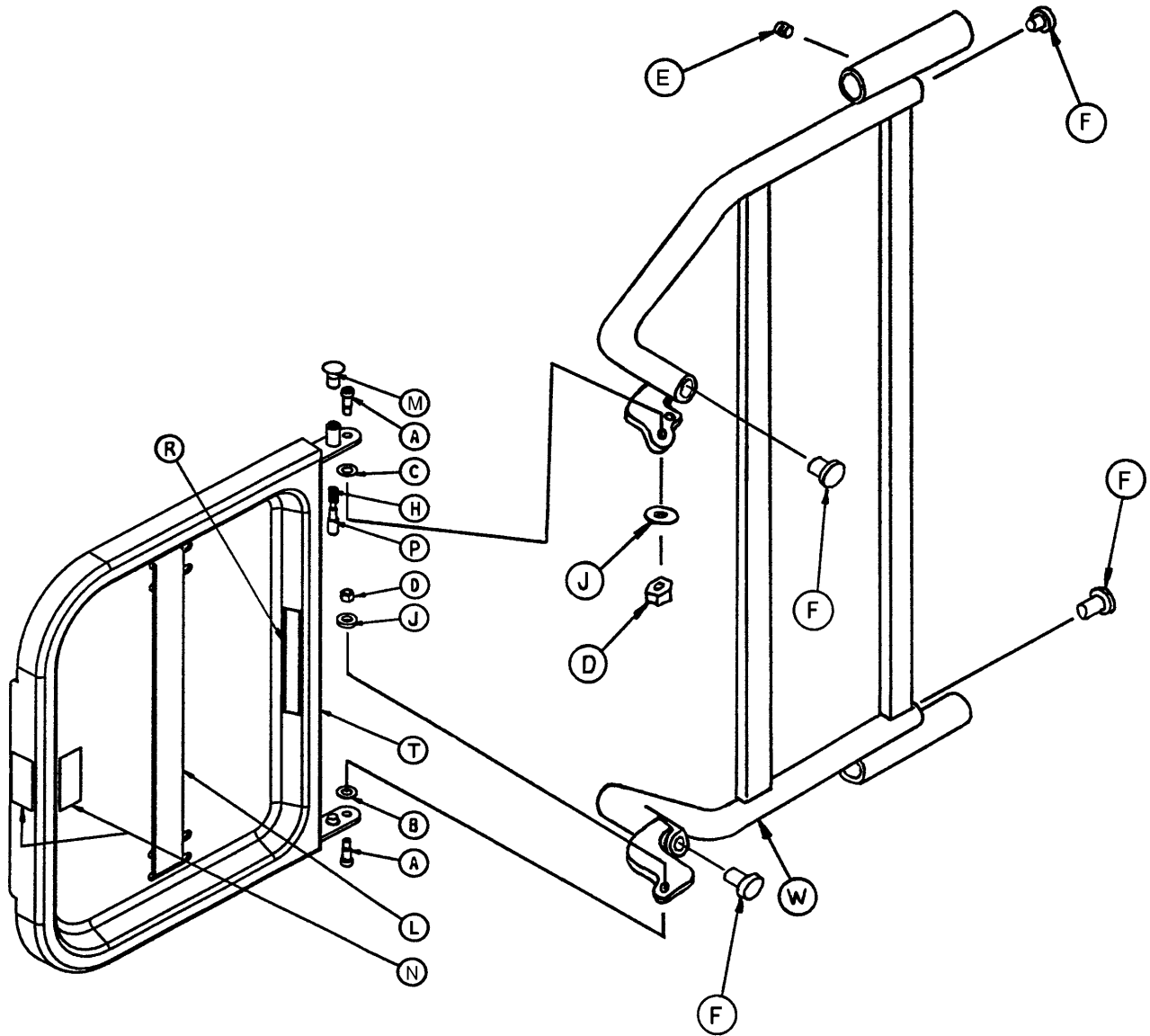

Item	Part No.	Part Name	Qty.	Item	Part No.	Part Name	Qty.
A	8-49	Soc. Hd. Shoulder Bolt	2	L	1010-50-21	Long Strap	1
B	14-20	Thrust Washer	1	M	1010-50-50	Knob	1
C	14-21	Thrust Washer	1	N	1010-50-57	Max. Weight Label	4
D	16-28	Fiberlock Nut	2	P	1010-50-242	Lock Pin	1
E	21-17	Set Screw	4	R	2025-120-5	Equipment Label	1
F	37-214	Hole Plug	4	S	2025-120-6	Specification Label	1
H	38-133	Spring	1	T	2025-120-18	Tray Assembly	1
J	52-17	Spacer	2	W	2025-120-25	Pivot Weldment Frame	1
K	1010-50-19	"Push/Pull" Label	1				

2025-40 Optional Bed Extender Pad Assembly

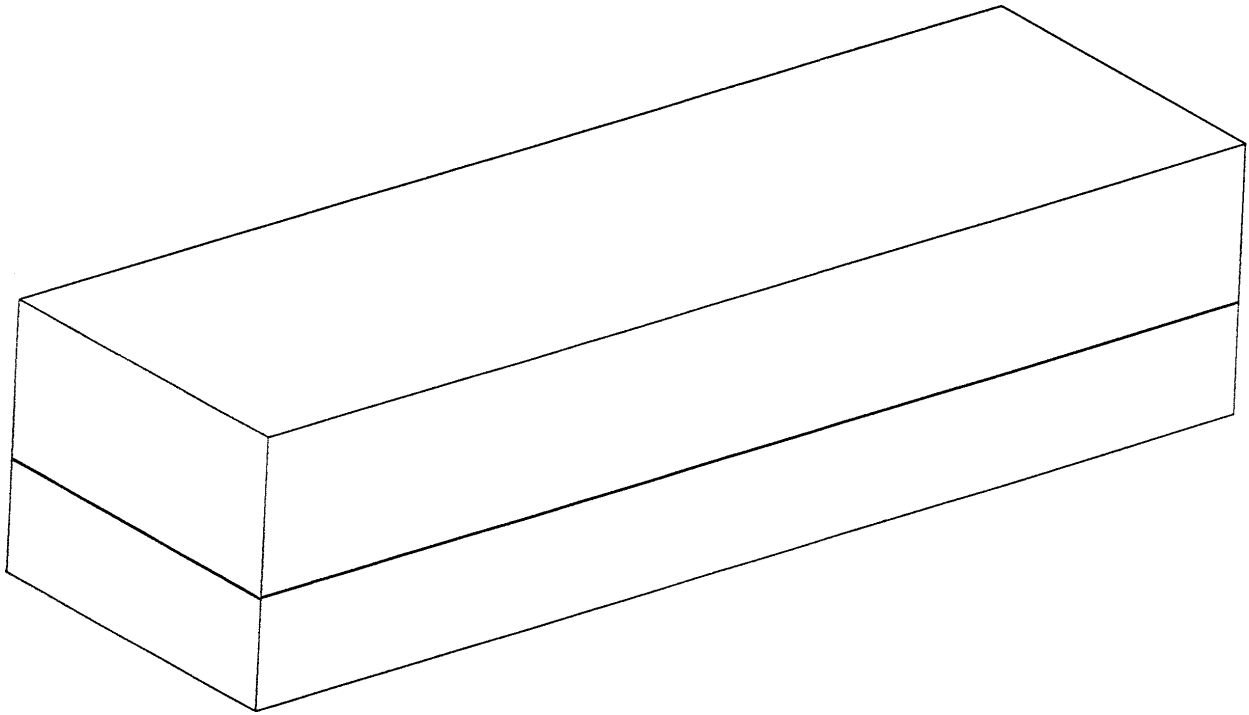



European Representative

Stryker France
BP 50040–95946 Roissy Ch. de Gaulle
Cedex–France

Phone: 33148632290
Fax: 33148632175

stryker[®]
Medical

6300 Sprinkle Road, Kalamazoo, MI 49001–9799 USA

(800) 327–0770



DH 5/00 2035–100–2 REV A