



Premier Bedside Stand, 3-Drawer - Model 4400-603-000

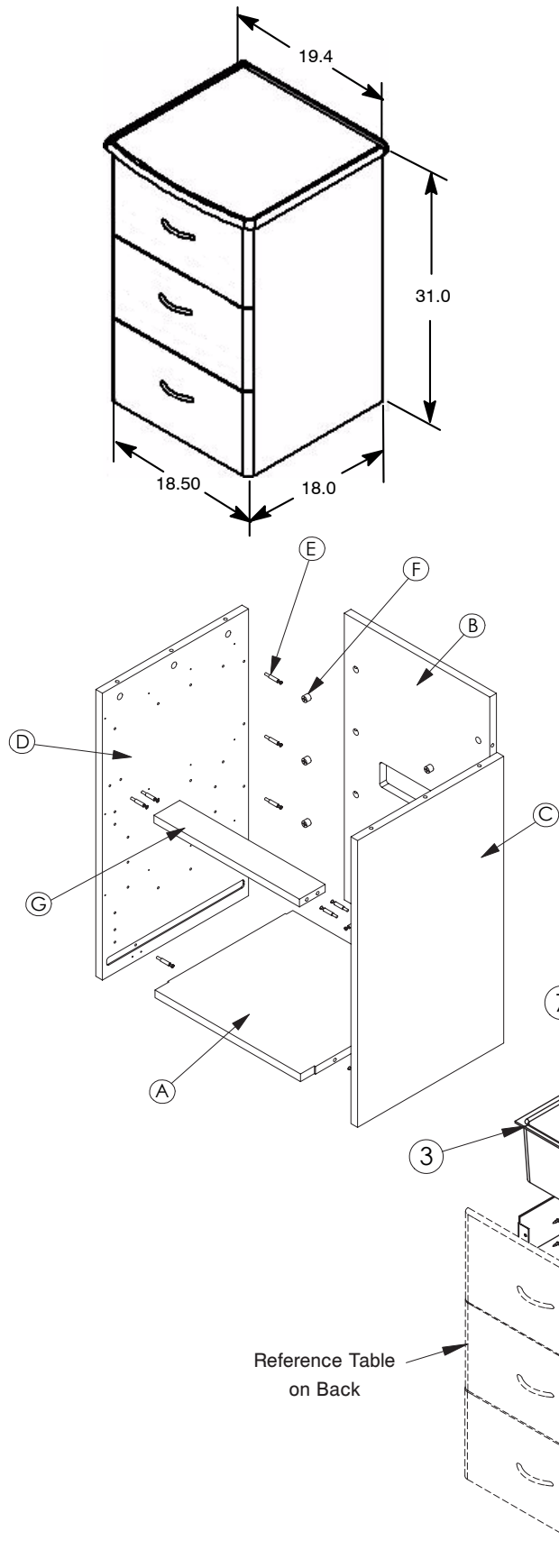

ASSEMBLY METHOD

The Stryker Med/Surg Beside Stand is constructed using a locking cam and dowel pin arrangement.

To dismantle the unit, use a #2 Phillips screwdriver to unlock each cam by rotating the cam in a counter-clockwise direction.

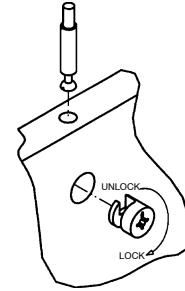

NOTE:

The product serial number label is located on the back of the unit.

Reference Table
on Back

Reference Table
on Back

stryker[®]

Premier Bedside Stand, 3-Drawer - Model 4400-603-000

Bedside Stand Common Components			
Item	Part Number	Part Name	Qty.
1	4400-508-100	Case (see Case Assembly Table)	1
3	3100-200-208	Drawer Liner	3
4	3150-001-200	Caster	4
5	3100-052-733	Dowel Pin	6
6	3100-052-732	Cam	6
7	0020-013-000	Wood Screw	12
8	4400-500-205	L-Bracket	3
9	0020-013-000	Wood Screw	64
10	4400-500-205	L-Bracket	3
11	3100-052-737	Plastic Vent	1
12	4400-600-203	Drawer Frame	3
13	3100-052-743	Rivet	18
14	4400-600-204	Drawer Slide (set of 2)	3
15	4400-009-100	Cleaning / Serial Number Label	1
16	0020-021-000	Wood Screw	28

(1) Case Assembly			
Item	Part Number	Part Name	Qty.
A	4400-508-118	Bottom Panel	1
B	4400-500-201	Back Panel	1
C	4400-500-216	Right Side Panel	1
D	4400-500-217	Left Side Panel	1
E	3100-052-733	Dowel Pin	16
F	3100-052-732	Cam	16
G	4400-500-204	Brace (No Lock)	1
	4400-500-224	Brace (Slam Lock)	1
	4400-500-223	Brace (Key Lock)	1

DISINFECTION PROCEDURE

The Center for Disease Control recommends a 1:4 mixture of bleach and water for disinfection. Thoroughly rinse the solution from the product surface. If other hospital-grade cleaning agents are used, follow the instructions provided by the cleaning solution's manufacturer. Wipe the surface with a soft, dry cloth to remove any moisture.

WARRANTY

Stryker warrants this product for one (1) year under normal use and proper care. For one year from date of shipment, Stryker will repair or, at our option, replace defective merchandise free of charge (parts, labor, and shipping included).

Note: LIFETIME warranty on metal drawer frames and glides.

3800 E. Centre Ave., Portage, MI 49002
(800) 327-0770 Option 2

Standard Configuration		
Part Number	Part Name	Qty.
4400-608-101	Drawer Front	1
4400-608-104	Cont. Pull Dwr. Front	1
4400-608-400	Door	1
4400-608-404	Continuous Pull Door	1
Top Panel Option		
Part Number	Part Name	Qty.
4400-200-210	Solid Spill Cont. Top	1
3100-200-210	Premier Top	1
Key Lock Option		
Part Number	Part Name	Qty.
4400-500-300	Lock	1
4400-608-102	Top Drawer Front	1
4400-608-101	Middle Drawer Front	1
4400-608-300	Bottom Drawer Front	1
Slam Lock Option		
Part Number	Part Name	Qty.
4400-500-301	Lock	1
4400-608-103	Top Drawer Front	1
4400-608-101	Middle Drawer Front	1
4400-608-300	Bottom Drawer Front	1
Key Lock Option with Continuos Pull		
Part Number	Part Name	Qty.
4400-500-300	Lock	1
4400-608-105	Top Drawer Front	1
4400-608-104	Middle Drawer Front	1
4400-608-304	Bottom Drawer Front	1
Slam Lock Option with Continuos Pull		
Part Number	Part Name	Qty.
4400-500-301	Lock	1
4400-608-106	Top Drawer Front	1
4400-608-104	Middle Drawer Front	1
4400-608-304	Bottom Drawer Front	1
Hardware		
Part Number	Part Name	Qty.
4400-600-206	Brass Pull	3
4400-600-207	Chrome Pull	3