
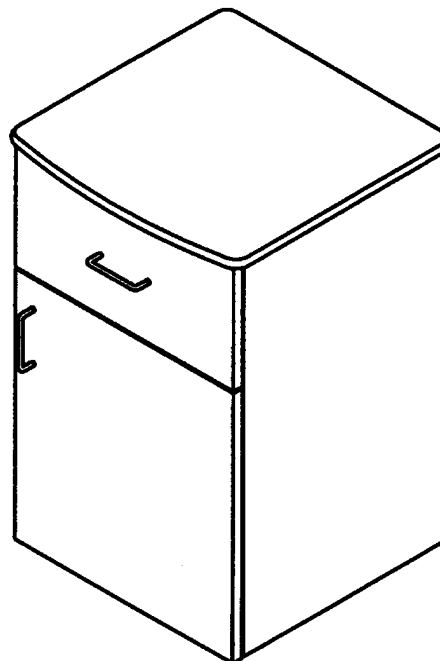


# LEGEND Bedside Stand (1-Drawer/1-Door) Model # 3100-200-100

## Operations and Maintenance

PRODUCT IDENTIFICATION LABEL  
(Located Underneath Unit)

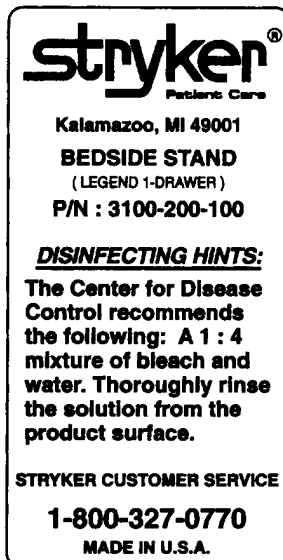

REPLACEMENT PARTS LIST		
Part Number	Part Name	
① 3100-200-110	Top Panel – Legend	
② 3100-200-130	Back Panel	
③ 3100-052-731	Plastic Drawer Slide (Pair)	
④ 3100-200-120	End Panel (Hinge Right)	
④ 3100-200-121	End Panel (No Hinge)	
⑤ 3100-200-140	Bottom Panel	
⑥ 3100-052-734	Twin Wheel Hooded Caster	
⑦ 3100-200-150	Front Panel – Hinge Right	
⑦ 3100-200-151	Front Panel – Hinge Left	
⑧ 3100-052-738	Pull w/Hardware – Legend (Chrome)	
⑧ 3100-052-735	Pull w/Hardware – Legend (Brass)	
⑨ 3100-200-108	6" Plastic Drawer	
⑩ 3100-200-122	End Panel (No Hinge)	
⑩ 3100-200-123	End Panel (Hinge Left)	
Warranty: 1 Year Parts and Labor		

### Assembly Method

The Select Series bedside stand is constructed using a locking cam and dowel pin arrangement (as shown in figure A). To dismantle the unit, use a #2 Phillips head screwdriver to unlock each cam by rotating the cam in a counter-clockwise direction.

### Replacement Parts

An exploded assembly view is provided on the opposite side of this card. Please reference the table shown to the left for parts shown in the exploded assembly view.

### Disinfection Procedure

The center for disease control recommends the following: A 1:4 mixture of bleach and water. Thoroughly rinse the solution from the product surface.

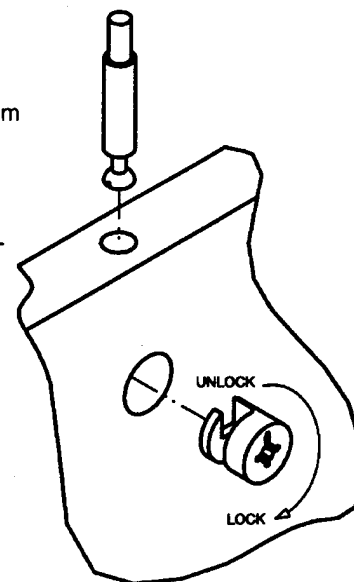


Figure A



Stryker Patient Care offers a full line of med/surg, maternity and critical care beds, furniture and pressure management products.

**LEGEND Bedside Stand (1-Drawer/1-Door) Model # 3100-200-100**

