Unity™ GC Belong™ Chair

Operations/Maintenance Manual

REF 3500

Symbols

	Consult instructions for use
	Caution
	Catalogue number
	Serial number
	Distributed by
	Manufacturer
	For US Patents see www.stryker.com/patents
	Safe working load

Table of Contents

Warning/Caution/Note Definition	2
Summary of safety precautions	2
Introduction	3
Product description	3
Intended use	3
Expected service life	3
Contact information	3
Serial number location	3
Specifications	5
Low back - single seat	5
Low back - single seat, 20 in. seat height	6
High back - single seat	7
High back - single seat, 20 in. seat height	8
Low back - bariatric	9
High back - bariatric	10
High back - flex back	11
High back - flex back, 20 in. seat height	12
Low back - single seat, armless	13
Accessories and parts	14
Preventive maintenance	15
Cleaning with SideKick® wipes	16
Cleaning the product	17
Service	18
Arm cap replacement	18
Seat cushion replacement	18
Back cushion replacement	19
Foot replacement	20
Low back - single seat	22
High back - bariatric	24
High back - flex back	26
Low back - single seat, armless	28

Warning/Caution/Note Definition

The words **WARNING**, **CAUTION**, and **NOTE** carry special meanings and should be carefully reviewed.

WARNING

Alerts the reader about a situation which, if not avoided, could result in death or serious injury. It may also describe potential serious adverse reactions and safety hazards.

CAUTION

Alerts the reader of a potentially hazardous situation which, if not avoided, may result in minor or moderate injury to the user or patient or damage to the product or other property. This includes special care necessary for the safe and effective use of the device and the care necessary to avoid damage to a device that may occur as a result of use or misuse.

Note - Provides special information to make maintenance easier or important instructions clearer.

Summary of safety precautions

CAUTION

- Improper usage of the product can cause injury to the patient or operator. Operate the product only as described in this manual.
 - Do not modify the product or any components of the product. Modifying the product can cause unpredictable operation resulting in injury to patient or operator. Modifying the product also voids its warranty.
 - Always confirm stable foot placement and back support or get assistance from another person to avoid the risk of injury or product damage.
-

Introduction

This manual assists you with the operation or maintenance of your Stryker product. Read this manual before operating or maintaining this product. Set methods and procedures to educate and train your staff on the safe operation or maintenance of this product.

CAUTION

- Improper usage of the product can cause injury to the patient or operator. Operate the product only as described in this manual.
 - Do not modify the product or any components of the product. Modifying the product can cause unpredictable operation resulting in injury to patient or operator. Modifying the product also voids its warranty.
-

Note

- This manual is a permanent part of the product and should remain with the product even if the product is sold.
- Stryker continually seeks advancements in product design and quality. This manual contains the most current product information available at the time of printing. There may be minor discrepancies between your product and this manual. If you have any questions, contact Stryker Customer Service or Technical Support at 1-800-327-0770.

Product description

The **GC Belong™** chairs are a line of furniture products for the healthcare setting. Each product functions as a padded seat.

Intended use

The **GC Belong** chairs are intended to be used as furniture within the healthcare setting.

Expected service life

The **GC Belong** chairs have a 10 year expected service life under normal use conditions and with appropriate periodic maintenance.

Contact information

Contact Stryker Customer Service or Technical Support at: 1-800-327-0770.

Stryker Medical
3800 E. Centre Avenue
Portage, MI 49002
USA

To view your operations or maintenance manual online, see <https://techweb.stryker.com/>.

Have the serial number (A) of your Stryker product available when calling Stryker Customer Service or Technical Support. Include the serial number in all written communication.

Serial number location

The serial number for your product is located under the seat, toward the back of the chair (A).

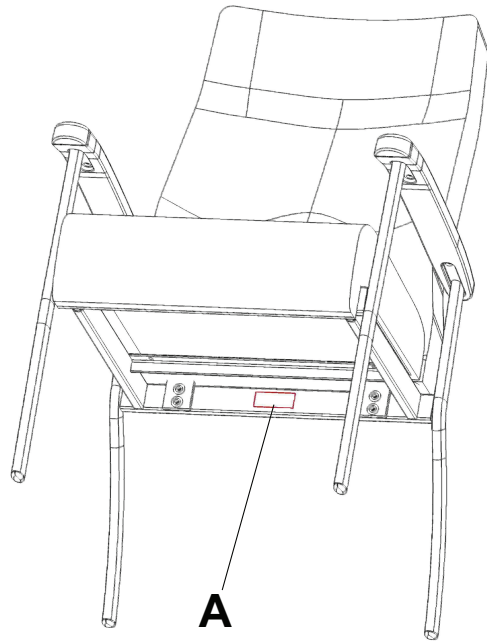
Figure 1 – Serial number location

Specifications

Low back - single seat

3500-000-501

	Safe working load	350 lb	158.8 kg
Overall length		28 in.	711 mm
Overall width		26.5 in.	673 mm
Overall height		37.5 in.	953 mm
Seat depth (minimum)		18 in.	457 mm
Seat width		22 in.	559 mm
Seat height (minimum)		18 in.	457 mm
Back width		21.5 in.	546 mm
Back height (minimum)		19.5 in.	495 mm
Arm height from seat (minimum)		7.5 in.	191 mm
Weight		41 lb	18.6 kg

Low back - single seat, 20 in. seat height

3500-000-502

	Safe working load	350 lb	158.8 kg
Overall length		28 in.	711 mm
Overall width		26.5 in.	673 mm
Overall height		39.5 in.	749 mm
Seat depth (minimum)		18 in.	457 mm
Seat width		22 in.	559 mm
Seat height (minimum)		20 in.	508 mm
Back width		21.5 in.	546 mm
Back height (minimum)		19.5 in.	495 mm
Arm height from seat (minimum)		7.5 in.	191 mm
Weight		42 lb	19 kg

High back - single seat

3500-000-503

	Safe working load	350 lb	158.8 kg
Overall length		28 in.	711 mm
Overall width		26.5 in.	673 mm
Overall height		43 in.	1092 mm
Seat depth (minimum)		18 in.	457 mm
Seat width		22 in.	559 mm
Seat height (minimum)		18 in.	457 mm
Back width		21.5 in.	546 mm
Back height (minimum)		24.5 in.	622 mm
Arm height from seat (minimum)		7.5 in.	191 mm
Weight		42.5 lb	19.3 kg

High back - single seat, 20 in. seat height

3500-000-504

	Safe working load	350 lb	158.8 kg
Overall length		28 in.	711 mm
Overall width		26.5 in.	673 mm
Overall height		45 in.	1143 mm
Seat depth (minimum)		18 in.	457 mm
Seat width		22 in.	559 mm
Seat height (minimum)		20 in.	508 mm
Back width		21.5 in.	546 mm
Back height (minimum)		24.5 in.	622 mm
Arm height from seat (minimum)		7.5 in.	191 mm
Weight		43.5 lb	19.7 kg

Low back - bariatric

3500-000-509

	Safe working load	750 lb	341 kg
Overall length		28 in.	711 mm
Overall width		37.5 in.	953 mm
Overall height		37.5 in.	953 mm
Seat depth (minimum)		18 in.	457 mm
Seat width		32.5 in.	826 mm
Seat height (minimum)		18 in.	457 mm
Back width		32.5 in.	826 mm
Back height (minimum)		19.5 in.	495 mm
Arm height from seat (minimum)		7.5 in.	191 mm
Weight		56 lb	25.5 kg

High back - bariatric

3500-000-510

	Safe working load	750 lb	341 kg
Overall length		28 in.	711 mm
Overall width		37.5 in.	953 mm
Overall height		43 in.	1092 mm
Seat depth (minimum)		18 in.	457 mm
Seat width		32.5 in.	826 mm
Seat height (minimum)		18 in.	457 mm
Back width		32.5 in.	826 mm
Back height (minimum)		24.5 in.	622 mm
Arm height from seat (minimum)		7.5 in.	191 mm
Weight		59 lb	26.8 kg

High back - flex back

3500-000-511

	Safe working load	350 lb	158.8 kg
Overall length		28 in.	711 mm
Overall width		26.5 in.	673 mm
Overall height		43 in.	1092 mm
Seat depth (minimum)		18 in.	457 mm
Seat width		22 in.	559 mm
Seat height (minimum)		18 in.	457 mm
Back width		21.5 in.	546 mm
Back height (minimum)		24.5 in.	622 mm
Arm height from seat (minimum)		7.5 in.	191 mm
Weight		48 lb	21.8 kg

High back - flex back, 20 in. seat height

3500-000-512

	Safe working load	350 lb	158.8 kg
Overall length		28 in.	711 mm
Overall width		26.5 in.	673 mm
Overall height		45 in.	1143 mm
Seat depth (minimum)		18 in.	457 mm
Seat width		22 in.	559 mm
Seat height (minimum)		20 in.	508 mm
Back width		21.5 in.	546 mm
Back height (minimum)		24.5 in.	622 mm
Arm height from seat (minimum)		7.5 in.	191 mm
Weight		49 lb.	22.2 kg

Low back - single seat, armless

3500-000-513

	Safe working load	350 lb	158.8 kg
Overall length		28 in.	711 mm
Overall width		24.75 in.	629 mm
Overall height		37.5 in.	953 mm
Seat depth (minimum)		18 in.	457 mm
Seat width		22 in.	559 mm
Seat height (minimum)		18 in.	457 mm
Back width		21.5 in.	546 mm
Back height (minimum)		19.5 in.	495 mm
Weight		33 lb	15 kg

Stryker reserves the right to change specifications without notice.

Accessories and parts

These accessories may be available for use with your product. Confirm availability for your configuration or region. Call Stryker Customer Service: 1-800-327-0770.

Name	Part number
Arm cap, black	3500-030-500
Arm cap, kalua	3500-031-500
Arm cap, latte	3500-032-500
Arm cap, mocha	3500-033-500
Arm cap, shadow	3500-034-500
Arm cap, toast	3500-035-500
Bolt, back	3500-039-500
Cushion, back, high	3500-024-500
Cushion, back, high, bariatric	3500-028-500
Cushion, back, high, flex	3500-025-500
Cushion, back, low	3500-026-500
Cushion, back, low, bariatric	3500-029-500
Cushion, seat	3500-023-500
Cushion, seat, bariatric	3500-027-500
Foot, back	3500-043-500
Foot, front	3500-042-500
Screw, arms, frame	3500-036-500
Screw, seat, wood	3500-100-079
Screw, seat, wood	3500-038-500
Split washer, back	3500-100-181
Washer, back	3500-041-500

Preventive maintenance

Remove product from service before you perform the preventive maintenance inspection. Check all items listed during annual preventive maintenance for all Stryker Medical products. You may need to perform preventive maintenance checks more often based on your level of product usage. Service only by qualified personnel.

Inspect the following items:

- _____ All fasteners are secure
- _____ Back, arms, and seat are not damaged
- _____ Feet are not damaged

Product serial number:
Completed by:
Date:

Cleaning with SideKick® wipes

For the United States only. Call Stryker Customer Service: 1-800-327-0770 to confirm availability for your configuration or region.

Stryker's preferred wipes (2060-000-001 6" x 10" or 2060-000-002 9" x 12") include the following active ingredients:

- n-Alkyl (60% C14, 30% C16, 5% C12, 5% C18) dimethyl benzyl ammonium chloride - 0.154%
- n-Alkyl (68% C12, 32% C14) dimethyl ethylbenzyl ammonium chloride - 0.154%
- Isopropanol - 21.000%

Non-active ingredient: Ethylene Glycol Monobutyl Ether – < 3%

Note - For safety information, read the product label.

Follow these steps to clean the external product surface using **SideKick** wipes. **SideKick** wipes are not approved to disinfect your product.

1. Rub with a fresh, clean **SideKick** wipe to remove all visible soils. Repeat as necessary until the product is clean.

Note - Use as many wipes as necessary.

2. Allow the external product surface to air dry before you return the product to service.

Cleaning the product

Recommended cleaners:

- Quaternary solutions (active ingredient – ammonium chloride) that contain less than 3% glycol ether
- Chlorinated bleach solution (1 part bleach solution to 100 parts water, 520 ppm available chlorine solution)
- Isopropyl alcohol (active ingredient \leq 21%)

Recommended cleaning method:

1. Follow the cleaning solution manufacturer's dilution recommendations.
2. Hand wash all surfaces of the product with a cleaning solution.
3. Avoid oversaturation. Do not allow the product to stay wet longer than the chemical manufacturer's guidelines for cleaning.
4. Dry the product.
5. Verify proper operation before you return the product to service.

Service

Arm cap replacement

Tools required:

- #3 Phillips screwdriver

Procedure:

1. Using a #3 Phillips screwdriver, remove the two screws (A) that secure the arm cap (B) to the chair frame (Figure 2). Remove and discard the arm cap. Save the screws.
2. Reverse steps to reinstall.
3. Verify proper operation before you return the product to service.

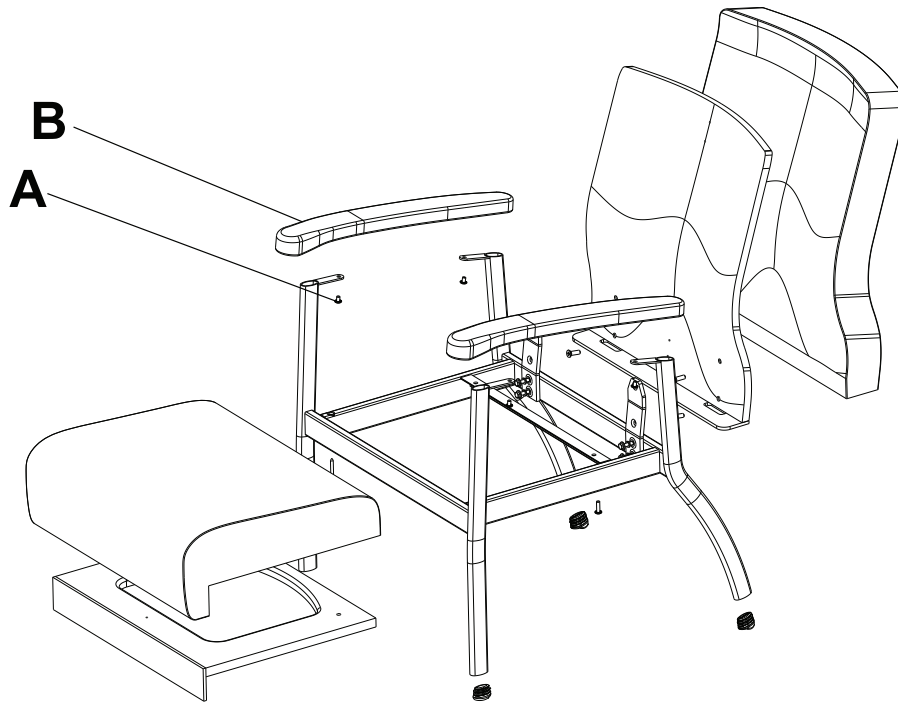
Figure 2 – Arm cap replacement

Seat cushion replacement

Tools required:

- #3 Phillips screwdriver
- Blanket or equivalent

Procedure:

1. Lay a blanket, or equivalent, on the floor behind the chair.
2. Using two hands, lift up on the bottom of the chair to flip the chair onto its back.

CAUTION - Always confirm stable foot placement and back support or get assistance from another person to avoid the risk of injury or product damage.

3. Using a #3 Phillips screwdriver, remove the four screws (A) that secure the seat cushion to the frame (Figure 3). Remove and discard the seat cushion. Save the screws.
4. Reverse steps to reinstall.

5. Verify proper operation before you return the product to service.

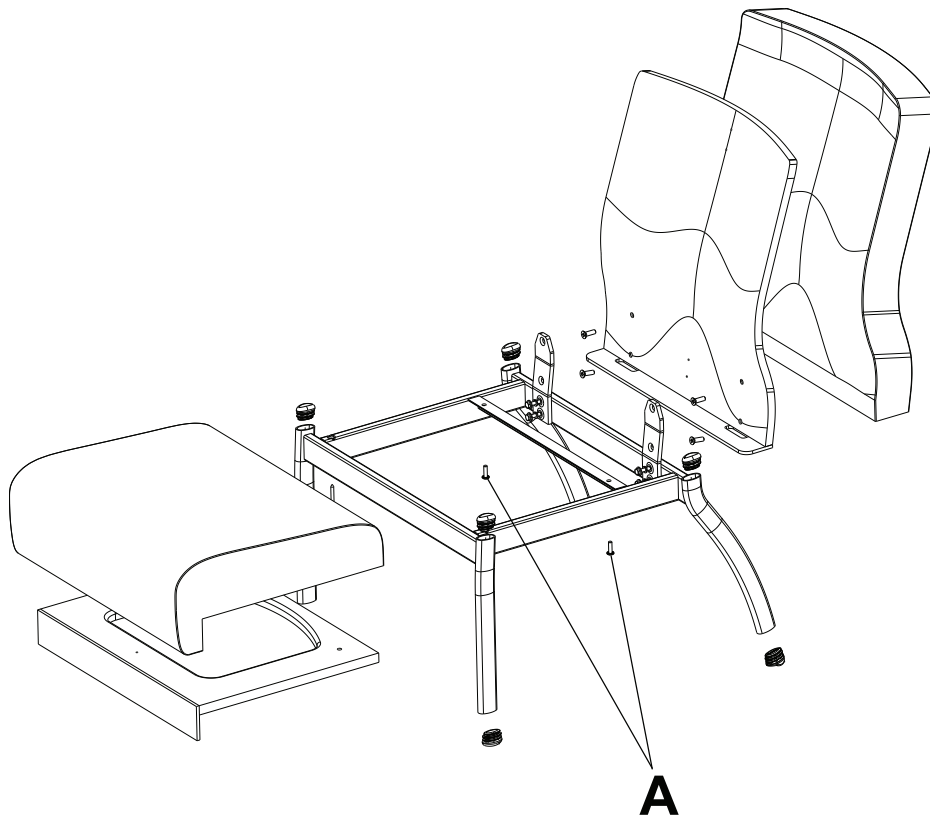
Figure 3 – Seat cushion replacement

Back cushion replacement

Tools required:

- 11mm combination wrench
- Blanket or equivalent

Procedure:

1. Lay a blanket, or equivalent, on the floor beside the chair.
2. Using two hands, flip the chair onto its side.
3. Using an 11mm combination wrench, remove the four bolts and washers (A) that secure the back cushion to the frame (Figure 4). Remove and discard the back cushion. Save the bolts and washers.
4. Reverse steps to reinstall.
5. Verify proper operation before you return the product to service.

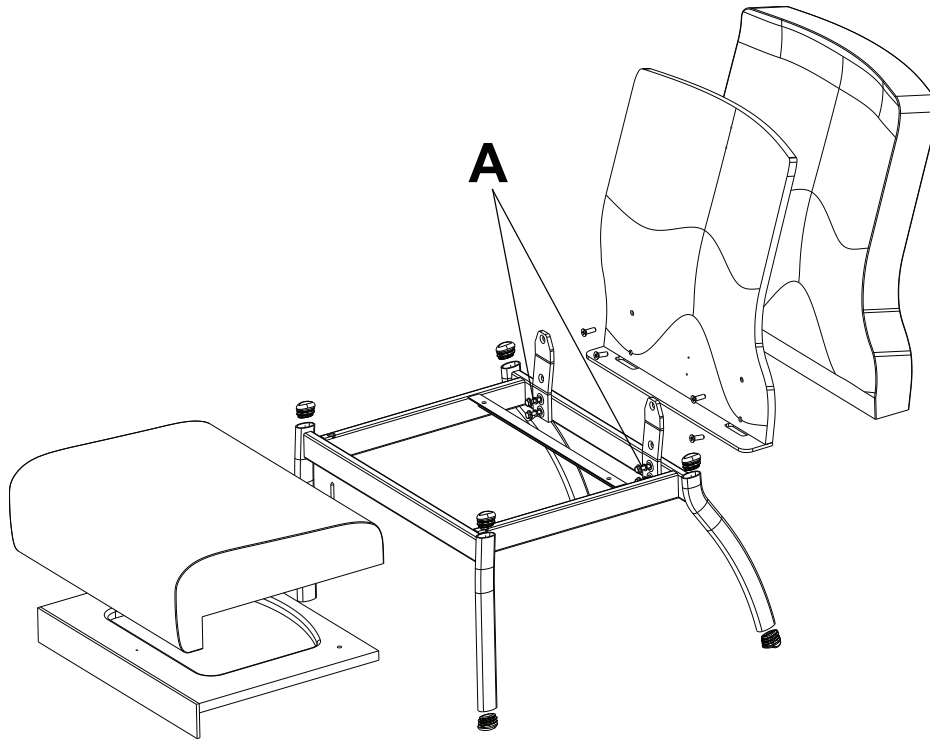
Figure 4 – Back cushion replacement

Foot replacement

Tools required:

- Flat screwdriver
- Rubber hammer
- Blanket or equivalent

Procedure:

1. Lay a blanket, or equivalent, on the floor beside the chair.

Note - Lay the blanket on the opposite side of the foot you are replacing.

2. Using two hands, flip the chair onto its side and onto the blanket.
3. Using a flat screwdriver and rubber hammer, drive the foot (A) out of the chair frame (Figure 5).
4. Using a rubber hammer, drive the new foot into the chair frame.

Note - Make sure that you use the proper foot replacement. The front and back feet are different. The back feet are direction specific.

5. Verify proper operation before you return the product to service.


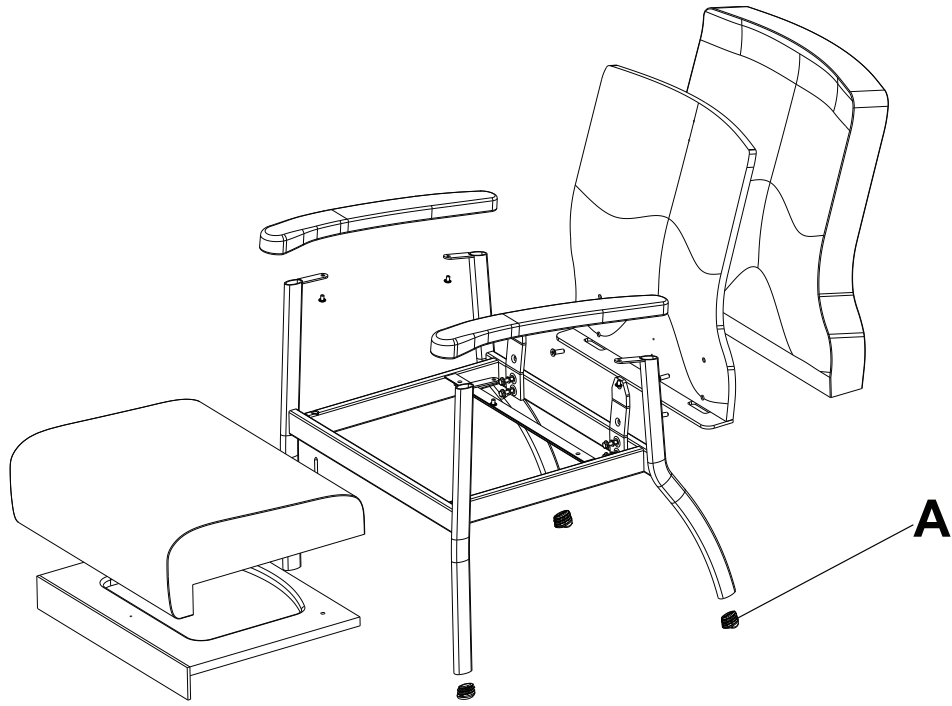
Figure 5 – Foot replacement

Low back - single seat

3500-000-501 (Reference only)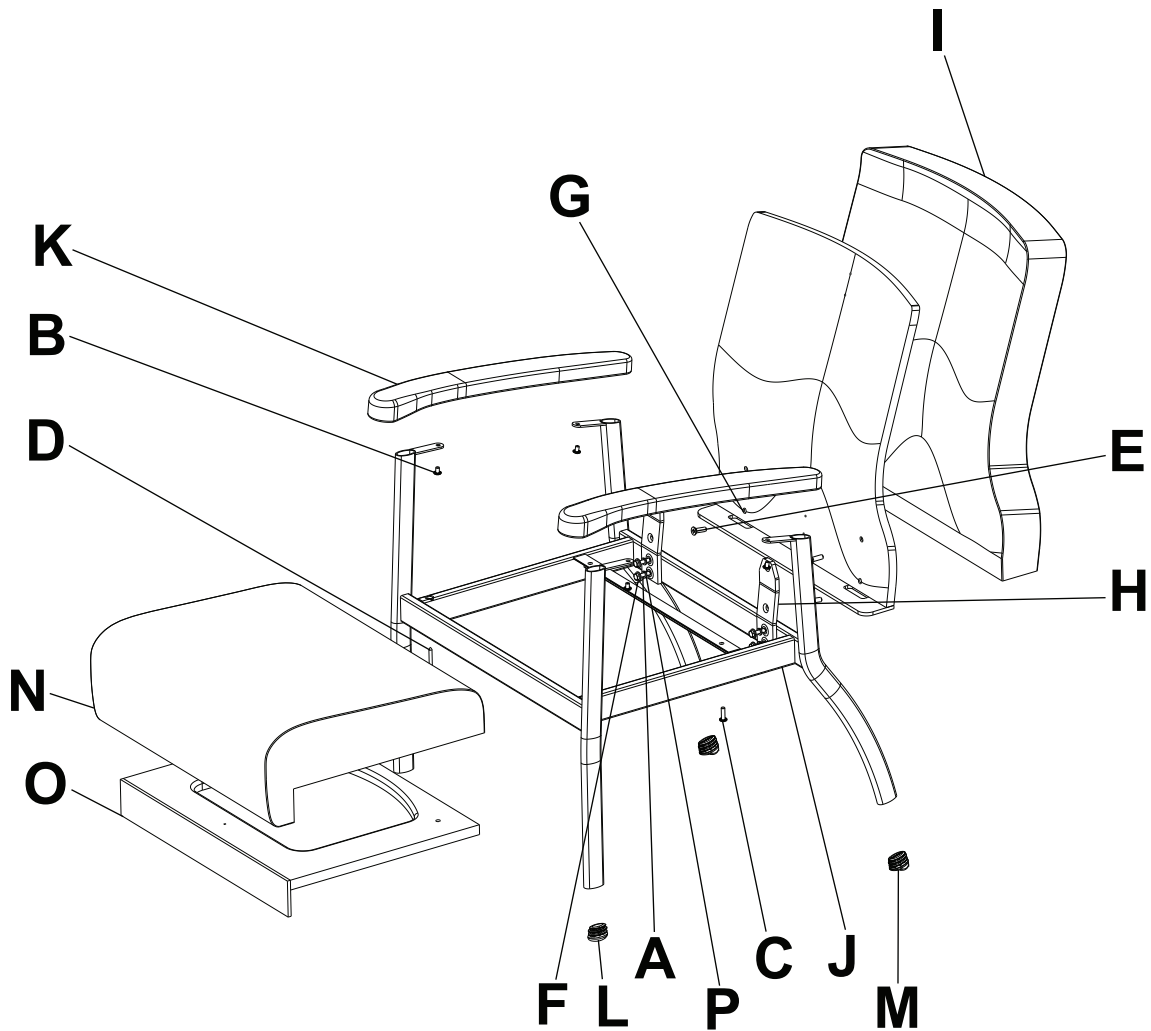


Item	Name	Quantity
A	Hex head bolt	4
B	Truss Phillips/square serrated screw, black	4
C	Truss Phillips/square serrated screw	2
D	Phillips/square wood screw	2
E	Phillips/square machine screw, black	4
F	Washer	4
G	Back wood with tee nut	1
H	Back bracket	2
I	Back foam	1
J	Chair frame, single seat	1
K	Arm cap with insert	2

Item	Name	Quantity
L	Foot, front	2
M	Foot, back	2
N	Seat foam	1
O	Seat wood	1
P	Washer, split lock	4

Note - Stryker Model 3500-000-502, 3500-000-503, and 3500-000-504 are not shown.

High back - bariatric

3500-000-510 (Reference only)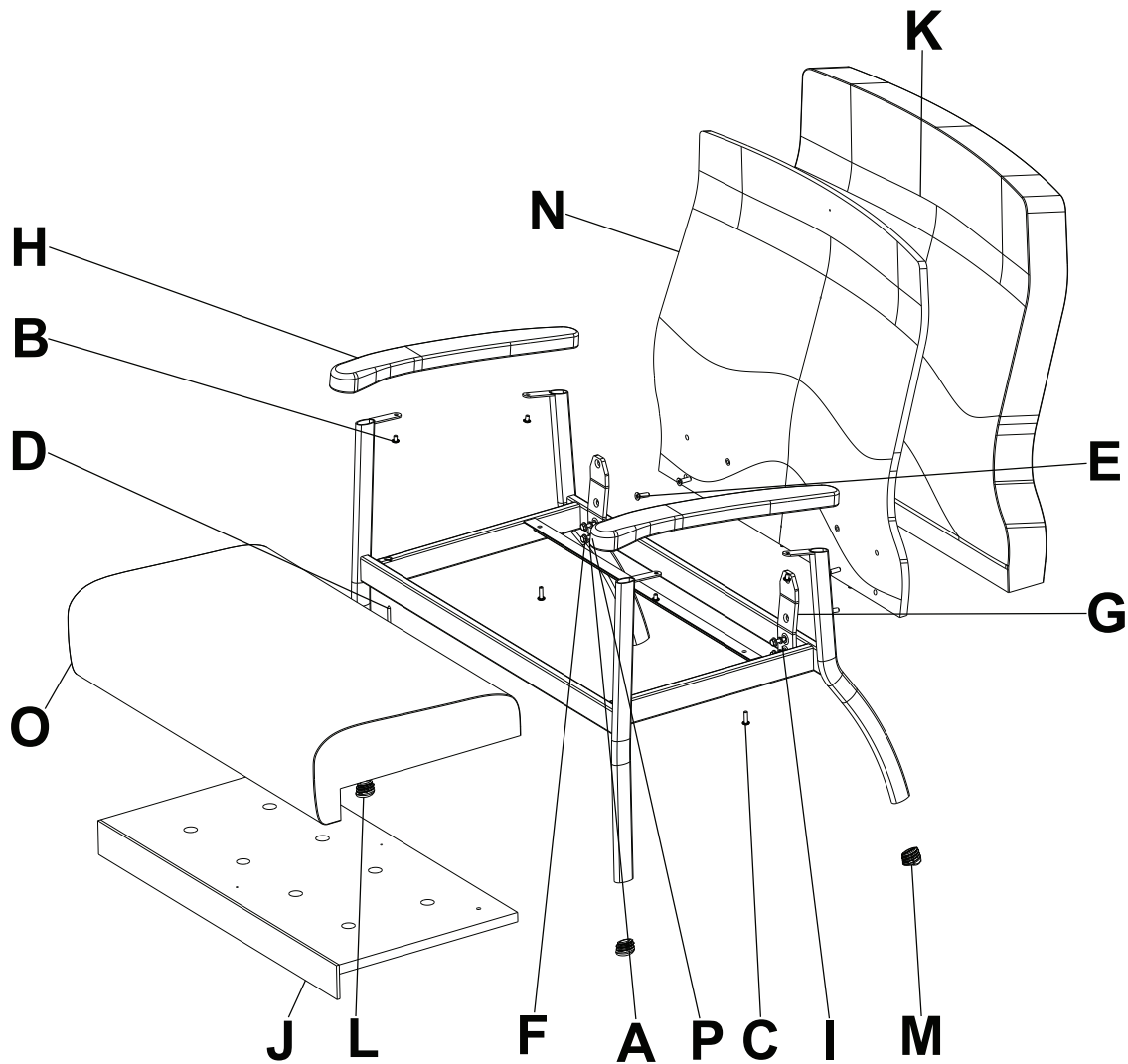


Item	Name	Quantity
A	Hex head bolt	4
B	Truss Phillips/square serrated screw, black	4
C	Truss Phillips/square serrated screw	2
D	Phillips/square wood screw	2
E	Phillips/square machine screw, black	4
F	Washer	4
G	Back bracket	2
H	Arm cap with insert	2
I	Chair frame, bariatric	1
J	Seat wood	1

Item	Name	Quantity
K	Back foam	1
L	Foot, front	2
M	Foot, back	2
N	Bariatric wood	1
O	Seat foam	1
P	Washer, split lock	4

Note - Stryker Model 3500-000-509 not shown.

High back - flex back

3500-000-511 (Reference only)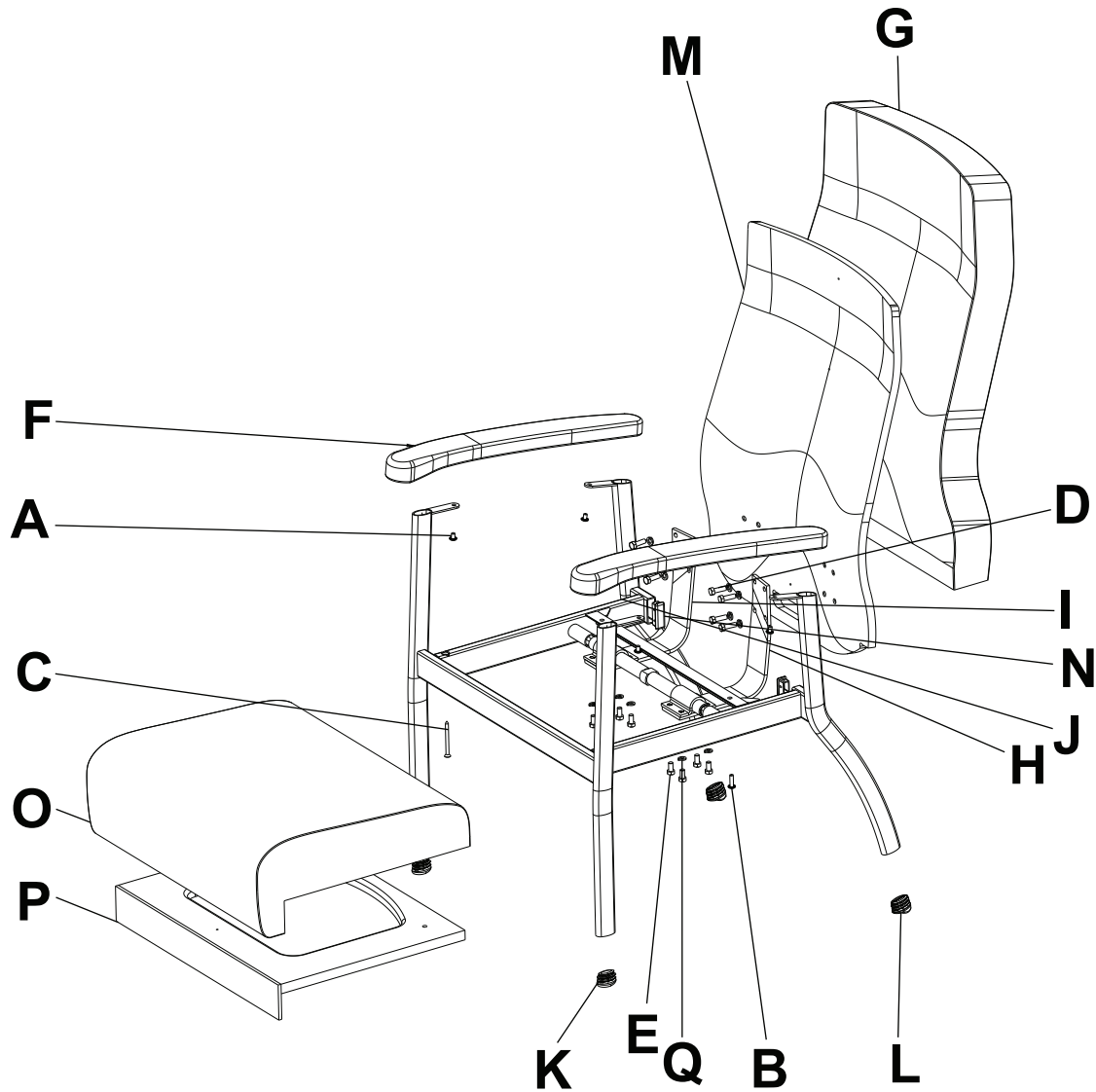


Item	Name	Quantity
A	Truss Phillips/square serrated screw, black	4
B	Truss Phillips/square serrated screw	2
C	Phillips/square wood screw	2
D	Hex head screw	8
E	Hex head screw	8
F	Arm cap with insert	2
G	Back foam	1
H	Back upright, left	1
I	Back upright, right	1

Item	Name	Quantity
J	Chair frame, flex back	1
K	Foot, front	2
L	Foot, back	2
M	Back wood with tee nut	1
N	Plastic closer	2
O	Seat foam	1
P	Seat wood	1
Q	Washer, split lock	16

Note - Stryker Model 3500-000-512 not shown.

Low back - single seat, armless

3500-000-513

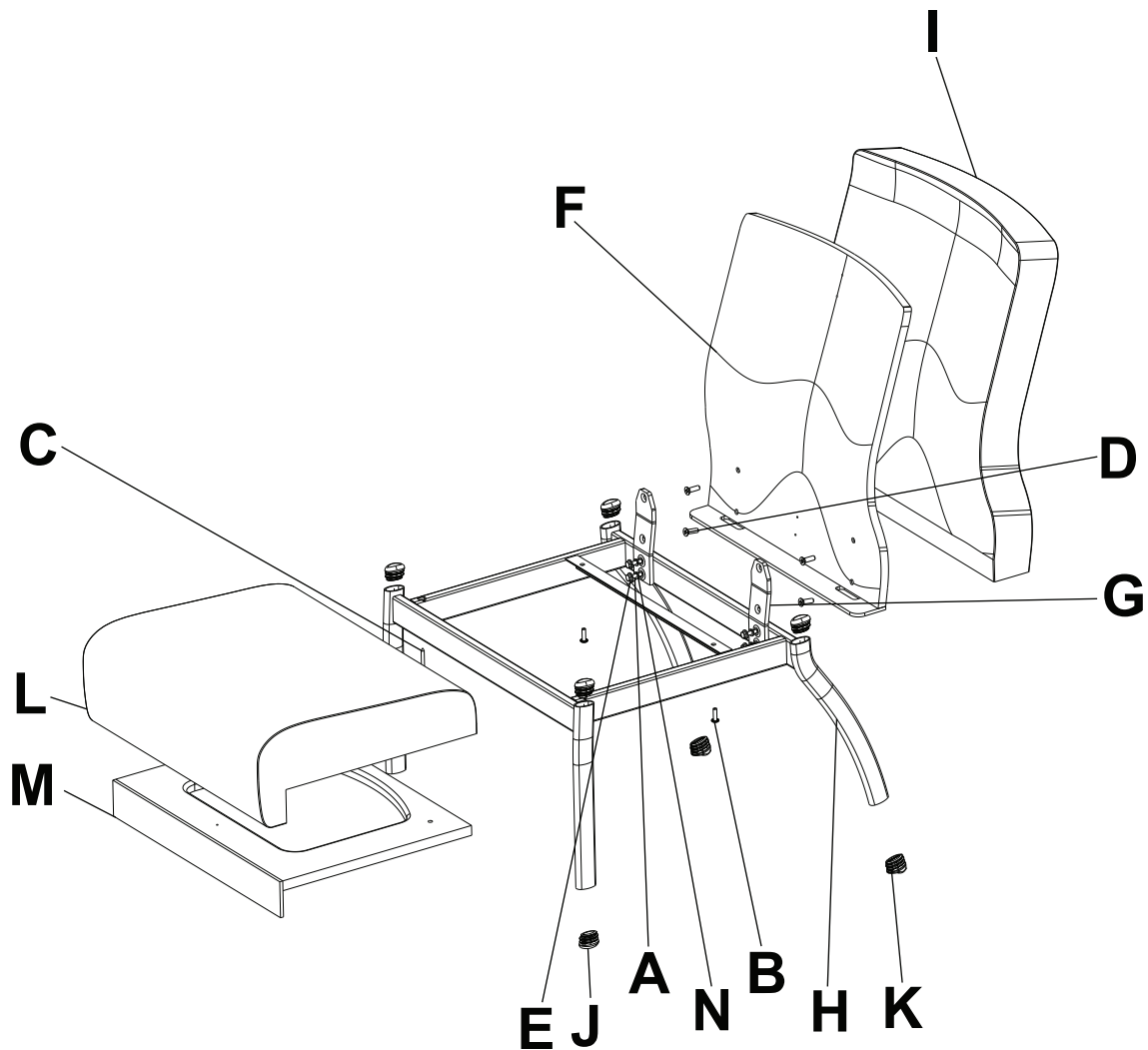Item	Name	Quantity
A	Hex head bolt	4
B	Truss Phillips/square serrated screw	2
C	Phillips/square wood screw	2
D	Phillips/square machine screw, black	4
E	Washer	4
F	Back wood with tee nut	1
G	Back bracket	2
H	Chair frame, single seat, armless	1
I	Back foam	1
J	Foot, front	6
K	Foot, back	2

Item	Name	Quantity
L	Seat foam	1
M	Seat wood	1
N	Washer, split lock	4

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