stryker

Unity™ Coast™ Glider

Operations/Maintenance Manual

REF 3500

Symbols

Ţ <u>i</u>	Consult instructions for use
\triangle	Caution
REF	Catalogue number
SN	Serial number
S	Distributed by
	Manufacturer
US Patents	www.stryker.com/patents
<u>^</u>	Safe working load

3507009001-A EN

Table of Contents

Warning/Caution/Note Definition	2
Summary of safety precautions	
Introduction	
Product description	
Intended use	3
Expected service life	
Specifications	3
Contact information	
Serial number location	4
Accessories and parts	5
Preventive maintenance	
Cleaning with SideKick® wipes	7
Cleaning the product	8
Service Servic	
Arm cap replacement	9
Seat cushion replacement	9
Back cushion replacement	
Glider assembly	12

Warning/Caution/Note Definition

The words WARNING, CAUTION, and NOTE carry special meanings and should be carefully reviewed.

WARNING

Alerts the reader about a situation which, if not avoided, could result in death or serious injury. It may also describe potential serious adverse reactions and safety hazards.

CAUTION

Alerts the reader of a potentially hazardous situation which, if not avoided, may result in minor or moderate injury to the user or patient or damage to the product or other property. This includes special care necessary for the safe and effective use of the device and the care necessary to avoid damage to a device that may occur as a result of use or misuse.

Note - Provides special information to make maintenance easier or important instructions clearer.

Summary of safety precautions

CAUTION

- Improper usage of the product can cause injury to the patient or operator. Operate the product only as described in this
 manual.
- Do not modify the product or any components of the product. Modifying the product can cause unpredictable operation resulting in injury to patient or operator. Modifying the product also voids its warranty.
- Always use two people when you lift or move the product. Always confirm stable foot placement and back support to avoid the risk of injury or product damage.

EN 2 3507009001-A

Introduction

This manual assists you with the operation or maintenance of your Stryker product. Read this manual before operating or maintaining this product. Set methods and procedures to educate and train your staff on the safe operation or maintenance of this product.

CAUTION

- Improper usage of the product can cause injury to the patient or operator. Operate the product only as described in this
 manual.
- Do not modify the product or any components of the product. Modifying the product can cause unpredictable operation resulting in injury to patient or operator. Modifying the product also voids its warranty.

Note

- This manual is a permanent part of the product and should remain with the product even if the product is sold.
- Stryker continually seeks advancements in product design and quality. This manual contains the most current product
 information available at the time of printing. There may be minor discrepancies between your product and this manual. If
 you have any questions, contact Stryker Customer Service or Technical Support at 1-800-327-0770.

Product description

The Coast™ Glider is a furniture product that functions as a padded glider seat.

Intended use

The Coast Glider is intended to be used as furniture within the healthcare setting.

Expected service life

The **Coast** Glider has a 10 year expected service life under normal use conditions and with appropriate periodic maintenance.

Specifications

Safe working load	350 lb	159 kg	
Overall length	32.5 in.	826 mm	
Overall width	30 in.	762 mm	
Overall height	43.5 in.	1105 mm	
Seat depth (minimum)	19.5 in.	495 mm	
Seat width	21.5 in.	546 mm	
Seat height (minimum)	17.5 in.	445 mm	
Back width	22 in.	559 mm	
Back height (minimum)	28 in.	711 mm	
Arm height from seat	8.5 in.	216 mm	
Weight	100 lb	45.5 kg	

3507009001-A 3 EN

Stryker reserves the right to change specifications without notice.

Contact information

Contact Stryker Customer Service or Technical Support at: 1-800-327-0770.

Stryker Medical 3800 E. Centre Avenue Portage, MI 49002 USA

To view your operations or maintenance manual online, see https://techweb.stryker.com/.

Have the serial number (A) of your Stryker product available when calling Stryker Customer Service or Technical Support. Include the serial number in all written communication.

Serial number location

The serial number for your product is located under the seat, toward the back of the glider (A).

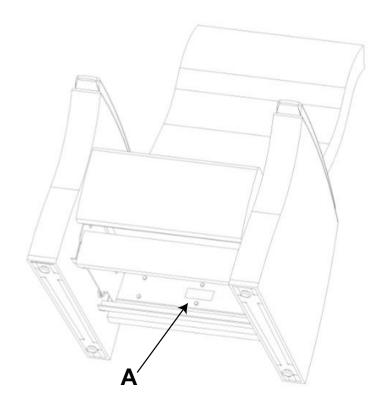
Figure 1 – Serial number location

EN 4 3507009001-A

Accessories and parts

These accessories may be available for use with your product. Confirm availability for your configuration or region. Call Stryker Customer Service: 1-800-327-0770.

Name	Part number
Arm cap, black	3500-008-700
Arm cap, kalua	3500-009-700
Arm cap, latte	3500-010-700
Arm cap. mocha	3500-011-700
Arm cap, shadow	3500-012-700
Arm cap, toast	3500-013-700
Bolt, back cushion	3500-039-500
Cushion, back	3500-018-700
Cushion, seat	3500-015-700
Foot, chair	3500-017-700
Screws, arm cap	3500-014-700
Screws, seat cushion	3500-100-077

3507009001-A 5 EN

Preventive maintenance

Remove product from service before you perform the preventive maintenance inspection. Check all items listed during annual preventive maintenance for all Stryker Medical products. You may need to perform preventive maintenance checks more often based on your level of product usage. Service only by qualified personnel.

Inspect the following items:
All fasteners are secure
Coast mechanism glides freely back and forth
Back, arms, and seat are not damaged
Product serial number:
Completed by:
Date:

EN 6 3507009001-A

Cleaning with SideKick® wipes

For the United States only. Call Stryker Customer Service: 1-800-327-0770 to confirm availability for your configuration or region.

Stryker's preferred wipes (2060-000-001 6" x 10" or 2060-000-002 9" x 12") include the following active ingredients:

- n-Alkyl (60% C14, 30% C16, 5% C12, 5% C18) dimethyl benzyl ammonium chloride 0.154%
- n-Alkyl (68% C12, 32% C14) dimethyl ethylbenzyl ammonium chloride 0.154%
- Isopropanol 21.000%

Non-active ingredient: Ethylene Glycol Monobutyl Ether – < 3%

Note - For safety information, read the product label.

Follow these steps to clean the external product surface using **SideKick** wipes. **SideKick** wipes are not approved to disinfect your product.

1. Rub with a fresh, clean **SideKick** wipe to remove all visible soils. Repeat as necessary until the product is clean.

Note - Use as many wipes as necessary.

2. Allow the external product surface to air dry before you return the product to service.

3507009001-A 7 EN

Cleaning the product

Recommended cleaners:

- Quaternary solutions (active ingredient ammonium chloride) that contain less than 3% glycol ether
- Chlorinated bleach solution (1 part bleach solution to 100 parts water, 520 ppm available chlorine solution)
- Isopropyl alcohol (active ingredient ≤ 21%)

Recommended cleaning method:

- 1. Follow the cleaning solution manufacturer's dilution recommendations.
- 2. Hand wash all surfaces of the product with a cleaning solution.
- 3. Avoid oversaturation. Do not allow the product to stay wet longer than the chemical manufacturer's guidelines for cleaning.
- 4. Dry the product.
- 5. Verify proper operation before you return the product to service.

EN 8 3507009001-A

Service

Arm cap replacement

Tools required:

- Rubber hammer
- · Large flat screwdriver

Procedure:

1. Using a rubber hammer, hit the back of the arm cap (A) toward the front of the glider until it stops moving forward (Figure 2).

Note - Use caution when you hit the arm cap to avoid the risk of damage to the fabric.

- 2. Position a large flat screwdriver between the arm cap and arm to pull up on the arm cap and loosen the arm cap from the glider. Remove and discard the arm cap.
- 3. Align the new arm cap bolts with the holes at the front of the arm slots.
- 4. Using a rubber hammer, hit the arm cap down until it is tight to the arm.
- 5. Using a rubber hammer, hit the arm cap toward the back of the glider until it stops moving back.

Note - Use caution when you hit the arm cap to avoid the risk of damage to the fabric.

6. Verify proper operation before you return the product to service.

Figure 2 - Arm cap

Seat cushion replacement

Tools required:

- · #3 Phillips screwdriver
- Blanket or equivalent

Procedure:

- 1. Lay a blanket, or equivalent, on the floor behind the glider.
- 2. Using two hands, lift up on the bottom of the glider to flip the glider onto its back.

CAUTION - Always use two people when you lift or move the product. Always confirm stable foot placement and back support to avoid the risk of injury or product damage.

3507009001-A 9 EN

- 3. Using a #3 Phillips screwdriver, remove the four screws (A) that secure the seat cushion to the glider frame (Figure 3). Remove and discard the seat cushion. Save the screws.
- 4. Reverse steps to reinstall.
- 5. Verify proper operation before you return the product to service.

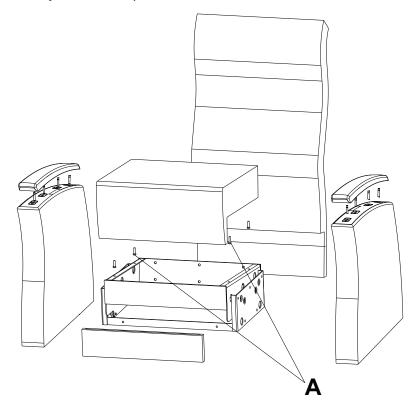
Figure 3 - Seat cushion replacement

Back cushion replacement

Tools required:

- 11mm combination wrench
- Blanket or equivalent

Procedure:

- 1. Lay a blanket, or equivalent, on the floor next to the glider.
- 2. Using two hands, lift up on the bottom of the glider to flip the glider onto its side and onto the blanket.

CAUTION - Always use two people when you lift or move the product. Always confirm stable foot placement and back support to avoid the risk of injury or product damage.

- 3. Using an 11mm combination wrench, remove the six bolts (A) that secure the back cushion to the glider frame (Figure 4). Remove and discard the back cushion. Save the bolts.
- 4. Reverse steps to reinstall.
- 5. Verify proper operation before you return the product to service.

EN 10 3507009001-A

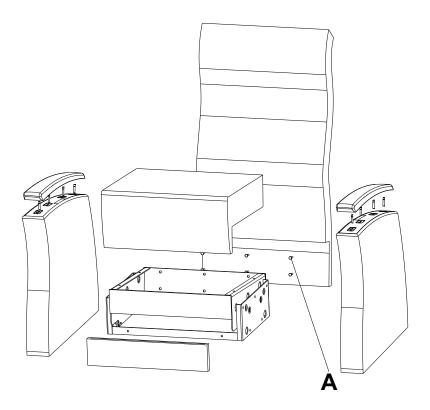
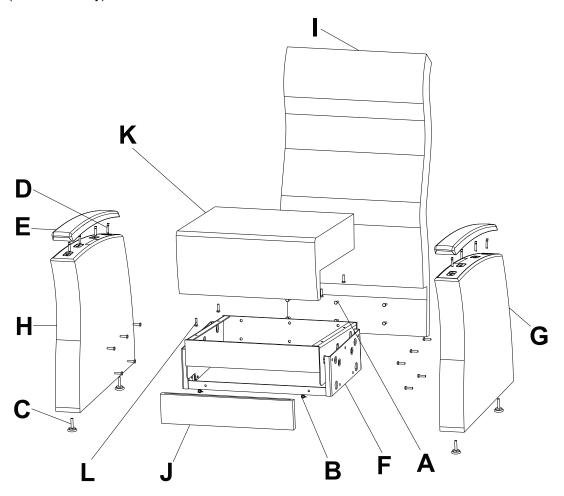
Figure 4 – Back cushion replacement

3507009001-A 11 EN

Glider assembly

3500-000-701 (Reference only)

Item	Name	Quantity
Α	Hex head bolt	16
В	Truss Phillips/square serrated screw, black	2
С	Adjustable glide	4
D	Double threaded stud	8
E	Arm cap with wood insert	2
F	Glider mechanism	1
G	Arm, upholstered, left	1
Н	Arm, upholstered, right	1
I	Back, upholstered	1
J	Kick plate, upholstered	1
K	Seat, upholstered	1
L	Truss Phillips/square serrated screw, black	4

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