HeelCare® Cushion

REF HC200CE

# stryker®

## Operations Manual

**Manuel d'utilisation**  
**Manuale d'uso**  
**Manual de uso**  
**Bedienungshandbuch**  
**Gebruikshandleiding**  
**Användarhandbok**  
**Betjeningsmanual**  
**Bruksanvisning**  
**Käyttöohjekirja**  
**Εγχειρίδιο λειτουργιών**  
**Manual de Operação**

CE

## Intended Use

The HeelCare® Cushion (HC200CE) is to be placed under the calves to provide relief of tissue interface pressures of the patients heels. The cushion is to assist in the treatment and prevention of pressure ulcers by floating both heels. Product is for single patient use.

## Symbols and Definitions

	Warning consult instructions for use		Single Patient Use
	Hand Wash Only		No Solvents
	No Steam / High Temperatures		European Representative
			No Sharp Objects
	Manufacturer		Model Number

## Specifications

Size (Unfolded)	34" x 17" x 3"	(86 cm x 43 cm x 8cm)
Material	Polyethylene (Nonwoven)	
Packaging	10 per carton	
Safe working load	200 lbs	90.7 kg
Expected Life	30 days or single patient use	
Ambient Temperature	60°F to 85°F (15.6°C to 29.4°C)	
Storage Temperature	20°F to 120°F (-6.7°C to 48.9°C)	

## Instructions for use

### WARNING

- To avoid the risk of patient injury, check skin frequently. Consult physician if any redness or skin break occurs.
- Risk of cross-contamination (e.g. Infection, allergic reaction). Product is for single patient use.

Figure 1

### Preparing the bed

1. Standing at the foot end of the bed, unroll the cushion ensuring easy access to the air valve (Figure 1).
2. Place the mesh under the mattress or overlay (Figure 2).
3. The cushion may be inflated with or without a patient on it.

Figure 2

## Inflating the Cushion

1. Insert the inflator into the air valve. Turn on the inflator for 60 seconds. Inflator automatically avoids putting too much air into the cushion (Figure 3). Refer to SC525/SC505 Operations Manual.
2. Remove inflator, turn it off, and tighten the cap.
3. Check to make sure that the patients heel is lifted off the bed as shown in Figure 4.

Figure 3

## Deflating the Cushion

### **⚠ WARNING**

Deflate the cushion before CPR.

1. Unscrew the cap.
2. For slow or partial deflations, insert pen point or other device into one of the four holes in the valve disc. Air will escape slowly (Figure 5).
3. For fast or complete deflation, pull tab to remove disc from the air valve (Figure 6).

Figure 4

## Caring for HeelCare Cushion

Clean cushion with soap and water. Wipe dry. Do not use solvents or alcohol. Allow product to dry thoroughly.

## Warranty

Stryker Medical warrants that the Stryker HeelCare Cushion model HC200CE should be free from manufacturing non-conformances that affect product performance and customer satisfaction for a period of thirty (30) days after date of delivery. Stryker's obligation under this warranty is expressly limited to supplying replacement parts.

Figure 5

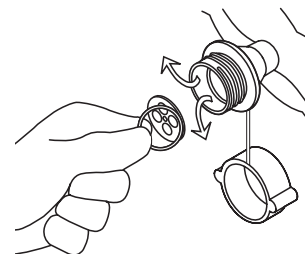
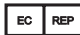
Figure 6

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**Stryker France S.A.S.**