

## **EFF302 ALTERNATING PAD**

### **Installation:**

- 1) Place the pad on top of the existing mattress, with tubing connectors at the foot end.
- 2) If desired, tuck the extension flaps under the existing mattress.
- 3) Follow PAR30 pump instructions for pump installation.
- 4) Connect one end of the tubing set to the pad connectors and the other end of the tubing to the pump connectors. Check to ensure that the tubing is not kinked or tucked under the mattress.

### **Cleaning:**

Clean by wiping the mattress with a damp cloth that is pre-soaked with warm water (below 40°C) and a mild detergent. A 10% Sodium Hypochlorite solution may also be used.

### **Storage:**

Lay the pad out flat and disconnect from the pump to deflate. Roll the pad from the head end towards the foot end and tubing connectors to remove excess air. The pad may also be rolled or folded.

### **Operating and Storage Environment:**

Operating Environment - 10°C to 40°C, 10% to 90% RH

Storage Environment - 0°C to 50°C, 10% to 90% RH

### **Cautions & Warnings:**

- Do not use Phenolic based products for cleaning.
- After cleaning, allow the pad to dry without direct exposure to sunlight.
- Product is for Single Patient Use only. Risk of cross-contamination if product were to be reused. (e.g. infection, allergic reaction)



Manufactured for:  
GAYMAR INDUSTRIES INC.  
10 Centre Drive  
Orchard Park, NY 14127  
Phone: 800-828-7341  
Fax: 800-993-7890