Sof•Care® BED CUSHIONS

REF SC402, SC440, SC460,  
SC427 and SC840

# stryker®

## Instructions for Use

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
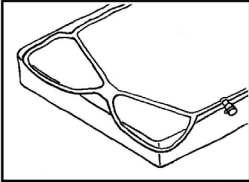
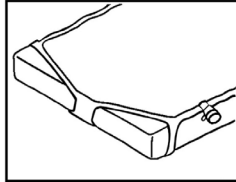
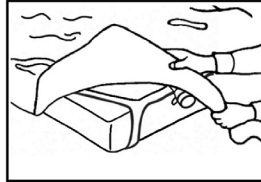
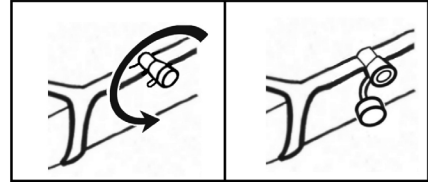
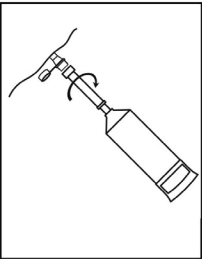
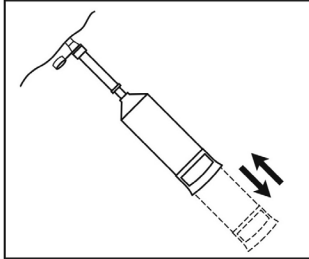
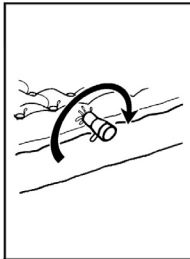
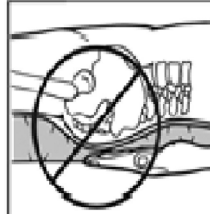
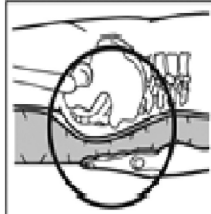
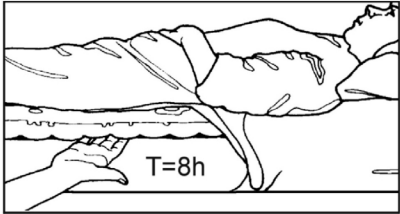
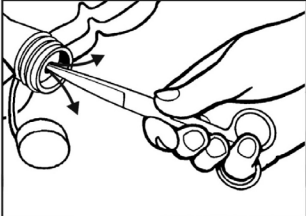
D3
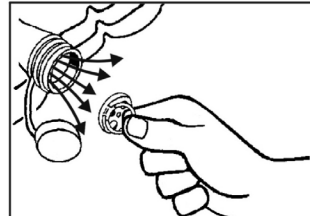
## Contraindications

Air support therapy is not recommended when spinal stability is a concern.

## Indications for Use

This device is intended to help treat and prevent decubitus ulcers (bedsores) and provide air support therapy.

Models SC402, SC440, SC460 and SC427 are for single patient use only. Do not reuse. Risk of cross-contamination (e.g. infection, allergic reaction) if product were to be reused.

Model SC840 is reusable.

## A. Preparing the bed

1. Standing at the foot end of the bed, unroll the cushion so that the air valve will be on the right side.
2. Place the corner straps around the bed mattress.
3. The bed cushion may be inflated with or without a patient on it. Place a sheet and incontinence pad (if required) over the cushion.
4. Unscrew the blue cap from the air valve.

## B. Inflating Sof•Care cushion with a HCP700 series manual inflator

1. Unscrew the standard cap and screw the inflator on to the air valve.
2. Manually inflate the cushion using the number of strokes indicated on the inflator (approximately 50 strokes for Sof•Care).
3. Unscrew the inflator and replace standard cap tightly.
4. Place sheet over cushion. Position patient on cushion.

**NOTE:** Overinflation may damage the bed cushion.

## C. Maintaining patient comfort and protection

The Sof•Care Bed Cushion normally has an underinflated appearance. It may seem to need more air when it really doesn't. The best way to determine if the cushion is properly inflated is by performing a HAND CHECK.

### To perform the HAND CHECK:

Place hand under the Bed Cushion, beneath the patient's buttocks. Hand must be positioned with palm up and fingers flat. (If fingers are flexed, the HAND CHECK will be misleading.) If patient does not touch hand, the Bed Cushion does not require reinflation. If patient does touch hand, reinflate using an Inflator or a manual pump and adapter (shown in step "D") until patient's buttocks no longer touch hand.

---

### CAUTION

Perform the HAND CHECK every 8 hours and after each inflation. Check skin frequently. Consult physician if any redness or skin break occurs. Serious injury could result if left untreated.

---

## D. Deflating Sof•Care Cushion

1. Unscrew the blue cap.
2. For slow or partial deflation, insert pen point or other device into one of the four holes in the valve disc. Air will escape slowly.
3. For fast or complete deflation, pull tab to remove disc from air valve.

---

### WARNING

Deflate before CPR or CPR will be ineffective.

---

## Caring for Sof•Care Cushions

Sof•Care cushions are easy to maintain. Perform the following care and cleaning to assure proper performance and reliability.

1. Deflate the Sof•Care cushion prior to cleaning. Clean cushion with a mild disinfectant or soap and water. Wipe dry. Do not use solvents or alcohol.
2. Note: Blood and other body fluids must be thoroughly cleaned from all surfaces before apply disinfectant.

Apply a disinfectant such as a 10% chlorinated bleach solution (chlorinated bleach with 5.25% sodium hydrochlorite) to the external surfaces of the overlay. Allow to completely dry. The solution contact time is what makes disinfection effective. Wipe down the overlay with a clean, dry cloth to remove excess disinfectant.

## Sof•Care Cushion Warranties

Standard Sof•Care cushions are warranted for 30 days. The Sof•Care Plus SC427 and SC427E are long term Sof•Care Bed Cushions and are warranted for 180 days. The Sof•Care Plus SC840 long term Sof•Care Bed Cushion is warranted for 2 years.

## Symbols and Definitions

	Warning consult instructions for use		Single Patient Use
	Caution		No Solvents
	Operating instructions		Hand Wash Only
	No Steam / High Temperatures		No Sharp Objects
	Foot end	<b>LOT</b>	Batch code
	Manufacturer	<b>REF</b>	Model Number
	European Representative		

## Specifications

<b>REF</b>	Size (Deflated)	Weight	Materials	Usage	Safe Working Load	Expected Life
SC402	74" x 33" (188cm x 83,8cm)	6 lb (2,7 kg)	Vinyl	Single Patient Use	350 lb (158.7 kg)	30 Days or SPU
SC427	40.5" x 91.4" (102,9 cm x 232,2 cm)	4.2 lb (1,9 kg)	Polyethylene EVA/ Nonwoven	Single Patient Use		180 Days or SPU
SC440	40.8" x 91.9" (103,6cm x 233,4cm)	2.4 lb (1,1 kg)	Polyethylene EVA/ Nonwoven	Single Patient Use		30 Days or SPU
SC460	40.8" x 91.9" (103,6cm x 233,4cm)	2.9 lb (1,3 kg)	Vinyl/ Nonwoven	Single Patient Use		30 Days or SPU
SC840	39" x 91" (99,1cm x 231,1cm)	4.9 lb (2,2 kg)	Polyethylene/ PVC Film	Reusable		2 years

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**Stryker France S.A.S.**