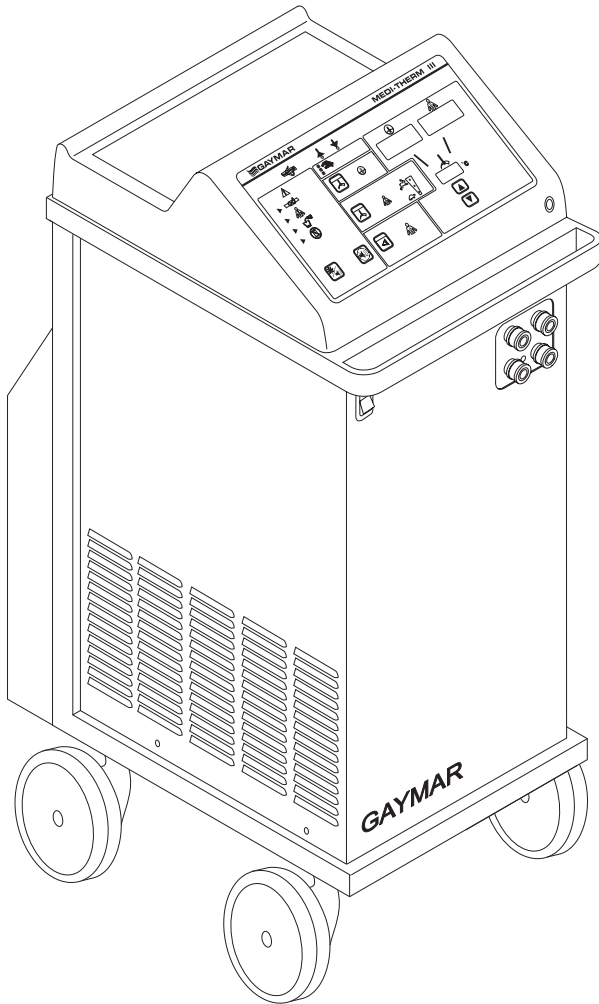
Medi-Therm® III

REF MTA7912

Gaymar®
Your patients. Our commitment.

<i>Operator's Manual</i>	[EN]
<i>Manuel de l'utilisateur</i>	[FR]
<i>Manuale per l'operatore</i>	[IT]
<i>Manual del operador</i>	[ES]
<i>Manual do Utilizador</i>	[PT]
<i>Bedienerhandbuch</i>	[DE]
<i>Bedieningshandleiding</i>	[NL]
<i>Brugerhåndbog</i>	[DA]
<i>Användarhandbok</i>	[SV]
<i>Brukerhåndbok</i>	[NO]
<i>Εγχειρίδιο χρήσης</i>	[EL]
<i>Návod k obsluze</i>	[CS]
<i>Felhasználói kézikönyv</i>	[HU]
<i>Instrukcja użytkownika</i>	[PL]
<i>Kullanım Kılavuzu</i>	[TR]

CE
0086

Table of Contents

Section	Description	Page
1.0	Indications for Use	1
2.0	Safety Precautions	1
3.0	Description	2
4.0	Operator Control Panel	4
5.0	Start-up Procedure	6
6.0	Modes of Operations:	
	BLANKET CONTROL	8
	PATIENT CONTROL.....	9
	MONITOR ONLY	10
7.0	Shutdown Procedure.....	10
8.0	Safety Systems	11
9.0	Troubleshooting.....	12
10.0	Care and Cleaning.....	14
11.0	Blankets/Body Wraps/Accessories ..	16
12.0	Customer Training.....	16
13.0	Warranty	16
14.0	Specifications	17

Illustrations

Figure	Description	Page
1	Medi-Therm III System.....	3
2	Medi-Therm III Control Panel.....	5
3	Float.....	6
4	Pinch Clamps	6
5	Quick-disconnects.....	7
6	Clik-Tite Connector	7
7	Probe Check Well	13
8	Probe Check Well	15

Receiving Inspection

Refer to the *Medi-Therm III Service Manual* for Receiving Inspection and Check-out Procedures.

If you have any questions, contact your local dealer.

Dangerous voltage

Year of manufacture

1.0 Indications for Use

For the prevention or treatment of hyperthermia/hypothermia.

1.1 Contraindications

Patients with an elevation of cold agglutinin levels (cold agglutininemia) should not be placed on a cooling blanket/body wrap.

In rare instances, the cold temperature of the cooling blanket/body wrap may lead to acrocyanosis and necrosis of distal body structures (Talisman, R., et al. *Surgery* 123(5): 592-595 (1998).

2.0 Safety Precautions

Read and understand this operating manual and all precautions listed below prior to using the Medi-Therm III Hyper/Hypothermia Machine.

⚠ WARNING

- If the patient's temperature is not responding or does not reach the prescribed temperature in the prescribed time or deviates from the prescribed temperature range, notify the attending physician promptly. **Failure to notify the physician promptly may result in serious injury or death.**
- When performing the *PROBE CHECK*, use a protective sheath on the probe. **Failure to use a sheath could result in cross-contamination.**
- A physician's order is required for use of equipment. Check the integrity of the skin according to department protocol when regulating temperature with external devices. Frequency of assessment and documentation will vary depending upon the individual response of the patient.
Failure to monitor patient may result in skin damage or inappropriate patient temperature.
- Avoid placing additional heat sources between the patient and blanket/body wrap. **Skin damage may result.**
- Use only Gaymar probes (see *Accessories*, page 16) or equivalent YSI 400 series probes approved for use with medical devices in the Patient Probe jack. **Use of other probes could result in erroneous patient temperature values.**
- Prevent excessive and/or prolonged tissue pressure and shearing forces, especially over bony prominences. **Skin damage may result.**

⚠ WARNING

- Keep the area between the patient and the blanket/body wrap dry. **Excessive moisture may result in skin damage.**
- Do not tip machine over without first draining the water out and unplugging the power cord. **Damage to the machine or electrical shock can result.**
- Use of thermally conductive materials (e.g., water or gel) on a patient when the Medi-Therm III is switched off may decrease the patient's body temperature.
- Repairs should be performed only by qualified medical service personnel in accordance with this product's service manual. **Otherwise, damage to the machine or improper therapy may result.**
- All wire-lead, patient-connected transducer assemblies are subject to reading error, local heating, and possible damage from high-intensity sources of RF energy. Inadequately grounded electrosurgical equipment represents one such source, since capacitively-coupled currents may seek alternate paths to ground through probe cables and associated instruments. **Patient burns may result.**

If possible, remove the probe from patient contact before activating the surgical unit or other RF source. If probes must be used simultaneously with electrosurgical apparatus, hazards can be reduced by selecting a temperature monitoring point which is remote from the expected RF current path to the ground return pad.
- When using the rate controlled Auto Moderate or Auto Gradual modes for warming; switching to other modes, e.g. Auto Rapid or Manual, or changing the temperature set point during the therapy will cause the Medi-Therm to reset the therapy. **Altering the Mode or Temperature set point may impact the overall duration of the therapy.**

⚠ CAUTION

- For grounding reliability, plug only into a properly grounded outlet.
- Fill with distilled water only. Do not use alcohol. Do not operate without water. Do not overfill. **Overfilling may result in overflow when the water in the blanket/body wrap drains back into the machine.**
- Place a dry absorbent sheet between the patient and the blanket when using the plastic-like side of any blanket. **Moisture may accumulate which may result in skin damage.**
- The *POWER LOSS INDICATOR* battery may discharge if the Medi-Therm III machine has not been used for two months or more. **The battery may not become recharged until the machine has operated for four hours.**

Hyperthermia/hypothermia blankets/body wraps may be placed either under, over, or around the patient, depending upon the type of procedure. Two gray hose sets are provided to connect multiple blankets and/or body wraps in order to provide more body surface contact. Increasing body surface contact facilitates more efficient warming/cooling.

NOTE: When connecting a second blanket/body wrap to the Medi-Therm III machine, check the water level (see *Start-up Procedure*, pp. 6-7) **prior** to connecting the additional blanket/body wrap.

In the event of a power loss, a battery backup circuit will light the *POWER LOSS* light and sound the audible alarm. Upon restoring power, the mode of operation and the set points will have to be reselected.

NOTE: Refer to the *MTA7912 Medi-Therm III Service Manual* for information on maintenance/replacement of the rechargeable battery.

3.0 Description

The MTA7912 provides a means of regulating patient temperature by supplying temperature-controlled water through a connector hose to a hyperthermia/hypothermia blanket/body wrap. The blanket/body wrap provides an interface for heating and cooling the patient. The machine controls the water temperature to a prescribed set point.

The feedback for control purposes depends upon the machine's operating mode. The machine may be operated in one of three operating modes:

In *BLANKET CONTROL* mode, the operator sets the desired water temperature. A temperature sensor within the machine monitors the water temperature and the machine heats or cools the water as required to bring the water to the *SET POINT* temperature. The patient temperature may be monitored by use of a patient probe connected to the patient probe jack on the front of the unit.

In *PATIENT CONTROL* mode, the Medi-Therm III machine automatically regulates the patient's temperature to the selected *SET POINT*. The machine constantly compares actual patient temperature with the *SET POINT* value, and automatically adjusts the water temperature so that the desired patient temperature is achieved.

In *MONITOR ONLY* mode, the operator can monitor patient temperature through the patient probe, without providing therapy.

The *MTA7912 Medi-Therm III Service Manual* contains circuit diagrams, functional test information, and component parts lists.

Water Fill Opening

Lift cover. Fill unit with distilled water until green band on float is fully visible (p. 6)

Patient Probe Jack

Accepts patient probe plug connecting patient probe to machine (pp. 8, 10). Use only Gaymar probes or equivalent YSI 400 series probes approved for use with medical devices.

Gaymar probe or equivalent YSI 400 series probe approved for use with medical devices.

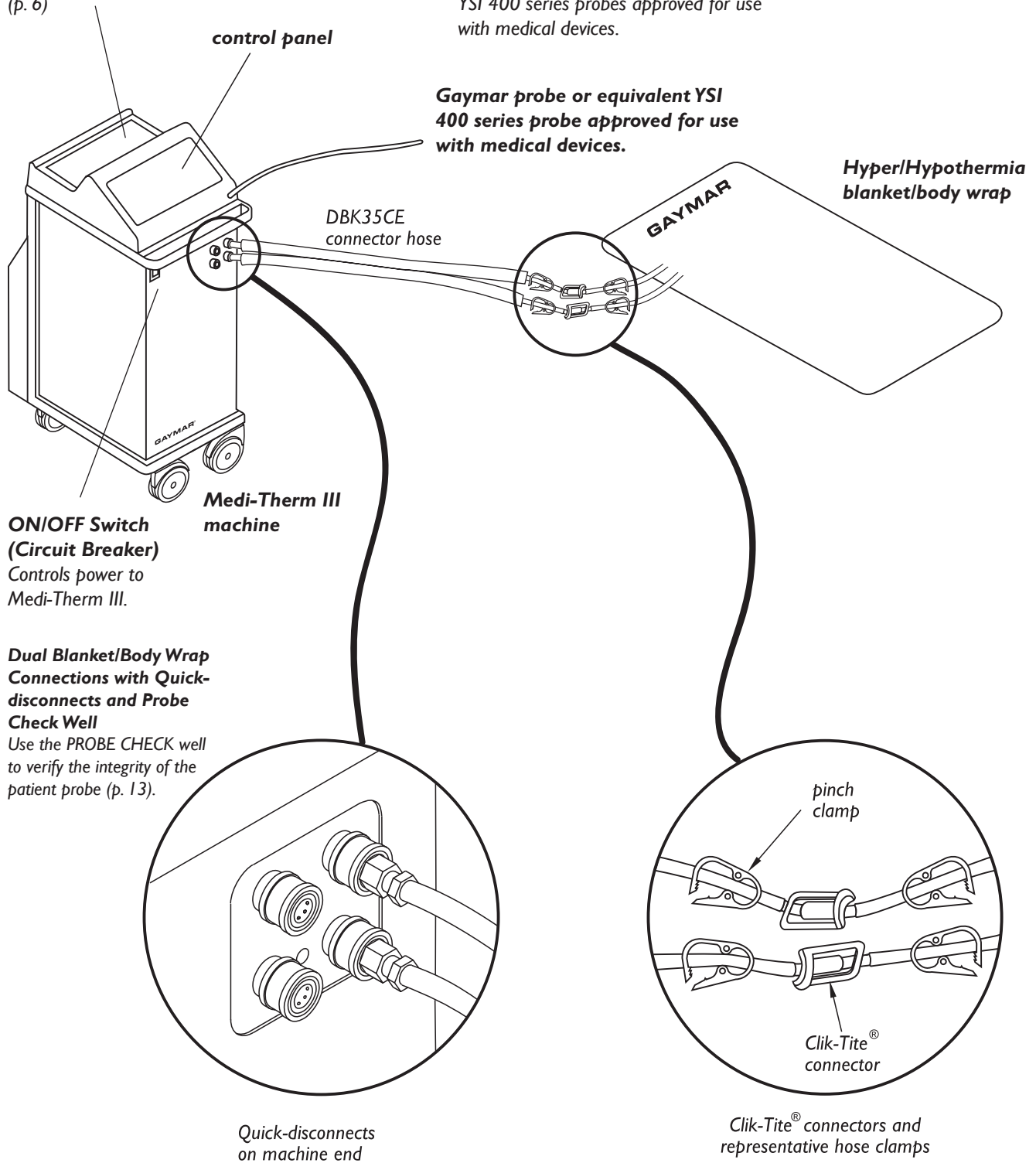
Figure 1—Medi-Therm III Hyper/Hypothermia System

4.0 Operator Control Panel

See figure 2, page 5.

MACHINE STATUS

FLOW-OK indicates water is flowing adequately.

WARMING indicates the machine is heating.

COOLING indicates the machine is cooling.

TEST LIGHTS flashes all indicator lights and digital displays, while sounding an audible alarm.

ALARM SILENCE temporarily silences the audible alarm. The audible alarm will reactivate in 10 minutes if the condition persists.

ALERTS

CHECK FLOW indicates the flow of water to the blanket/body wrap has been restricted.

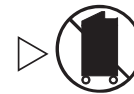
NOTE: When two gray hose sets are in use, the *CHECK FLOW* alarm will only sound if the flow through BOTH gray hose sets is occluded. (See page 7, section 5.6).

CHECK PATIENT indicates the patient probe is sensing an abnormal patient temperature (below 29°C or above 40°C). It may indicate the patient probe has become dislodged, an incorrect probe is being used, or a probe has become defective. An audible alarm accompanies this alert.


ADD WATER indicates the water reservoir is filled to less than the recommended level.

REMOVE FROM USE NOW indicates the Medi-Therm III machine has shut down due to a malfunction. **Remove the machine from use. Contact qualified medical service personnel.**

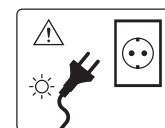FLASHING DISPLAY indicates that the water or patient temperature is not being maintained within the acceptable limits of the set point temperature. It is accompanied by an audible alarm. See pp. 8-10, sections 6.1 and 6.2 for explanations of the various alarms that have a flashing temperature display.

POWER LOSS INDICATOR lights when power is removed and *ON/OFF* circuit breaker is *ON*. [located on front of machine]



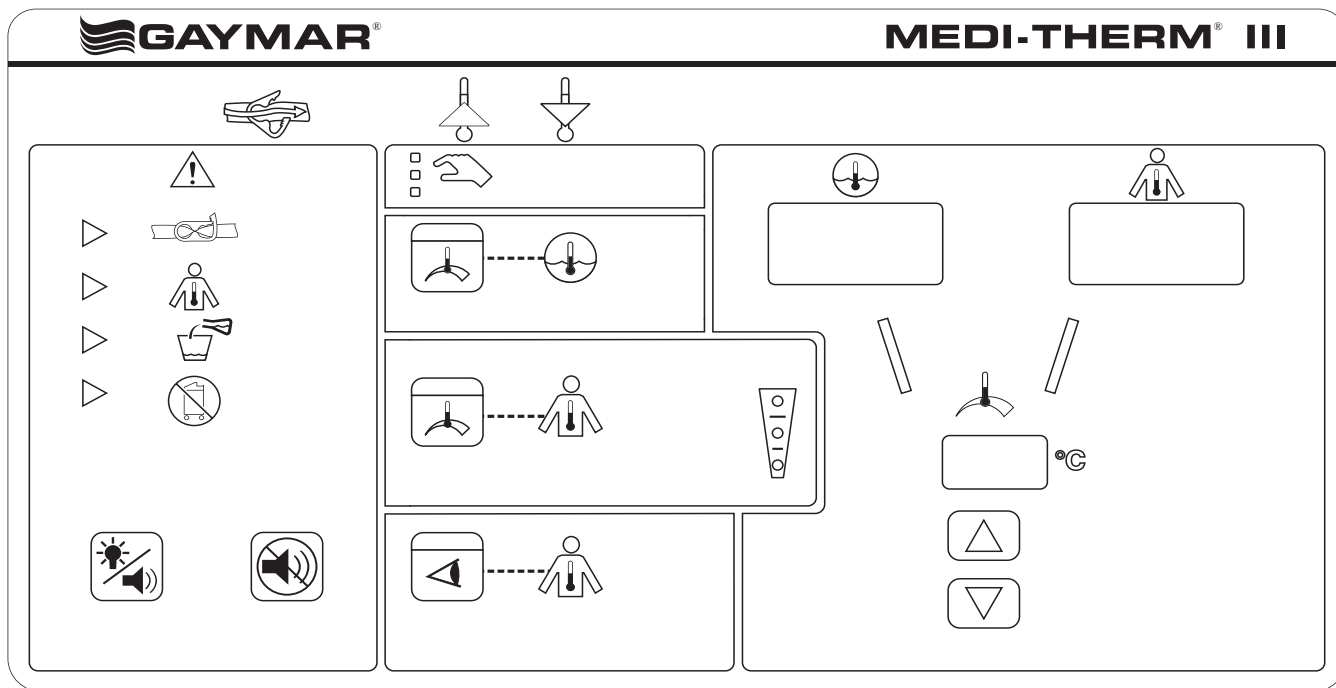
Figure 2—Medi-Therm III Control Panel

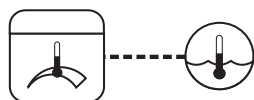
SELECT MODE

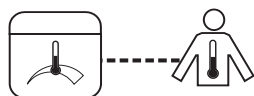
BLANKET CONTROL—
Adjust *SET POINT* to control water temperature. (4°C to 41°C)

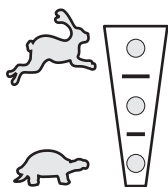PATIENT CONTROL—
Connect patient probe, then adjust *SET POINT* to control patient temperature. (30°C to 39°C)

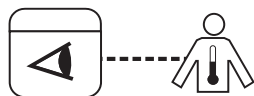CONTROL OPTION—
Cooling/Warming adjusted to *RAPID* rate.
Cooling/Warming adjusted to *MODERATE* rate.
Cooling/Warming adjusted to *GRADUAL* rate.

MONITOR ONLY—
Connect patient probe, then use to monitor patient temperature. No therapy is provided.

TEMPERATURE

WATER TEMPERATURE—
This display lights when either *PATIENT CONTROL* or *BLANKET CONTROL* is selected.

PATIENT TEMPERATURE—
As measured by the patient probe. This display lights whenever a patient probe is plugged in.

SET POINT CHANGE—
Press the “▼” or “▲” button to select machine set point. When selecting a temperature outside the normothermic range (36°C to 38°C), an audible alarm will chime. Press and hold the “▼” or “▲” button until the chime stops to access the extended temperature range.

PATIENT PROBE JACK—
Insert only a Gaymar probe or an equivalent YSI 400 series probe approved for use with medical devices. [located on front of machine]

5.0 Start-up Procedure

⚠ CAUTION

The *POWER LOSS INDICATOR* battery may discharge if the Medi-Therm III machine has not been used for two months or more. **The battery may not become recharged until the machine has operated for four hours.**

Perform the following start-up procedure each time the Medi-Therm III system is used:

5.1 Check Water Level

Check to make sure water is at the proper level by raising the cover on the water fill opening. Fill with distilled water until the green band on the float is fully visible (fig. 3).

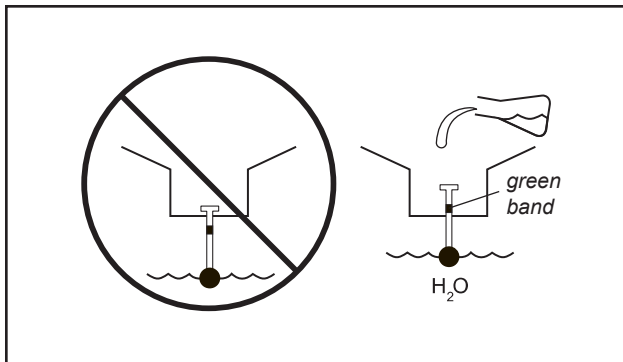
Figure 3—Float, showing proper water level

⚠ CAUTION

- **Do not operate without water**, since damage to internal components may result.
- **Do not overfill.** Overfilling may result in overflow because the water in the blanket/body wrap drains back into the machine when the machine is turned off.

5.2 Close Pinch/Slide Clamps

Pinch/slide clamps are provided on both the DBK35CE Connector Hose and the blanket/body wrap hoses to prevent water spillage when blankets/body wraps are changed (fig. 4).

Close the pinch/slide clamps on the connector hose and Hyper/Hypothermia blanket/body wrap.

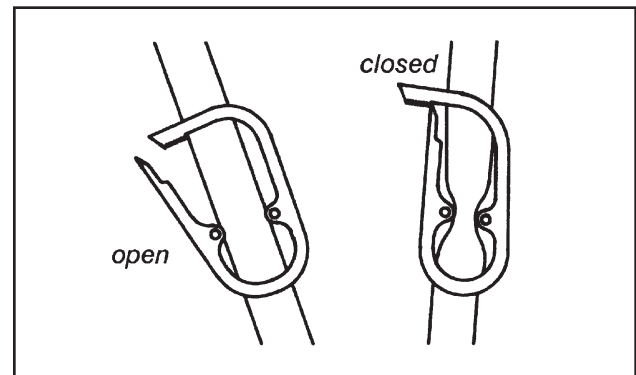
Figure 4—Pinch Clamps

5.3 Attach Blanket/Body Wrap

⚠ CAUTION

Check the surface of reusable blankets prior to each use. If the blanket shows signs of cracking or begins to leak, remove it from use and replace the blanket.

Attach the Hyper/Hypothermia blanket/body wrap to the Medi-Therm III machine using a DBK35CE Connector Hose (fig. 1, p. 3):

- I. Attach one quick-disconnect fitting to the **upper** machine connection and one to the **lower** (fig. 5, p. 7). Push each retaining collar toward the cabinet and insert the quick-disconnect fitting firmly. While holding the fitting, allow the retaining collar to snap back into position. Check to make sure the fitting is fully engaged by tugging firmly on the hose.

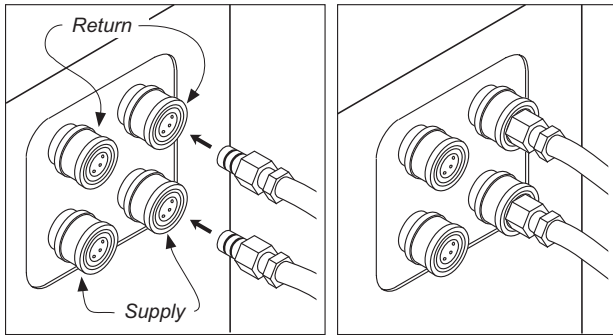
Figure 5—Quick-disconnects

2. Attach the two Clik-Tite connectors (fig. 6 below) on the connector hose to the blanket/body wrap. Insert the male fitting into the female. When fittings are fully seated, snap the locking ring into place to assure a positive lock.

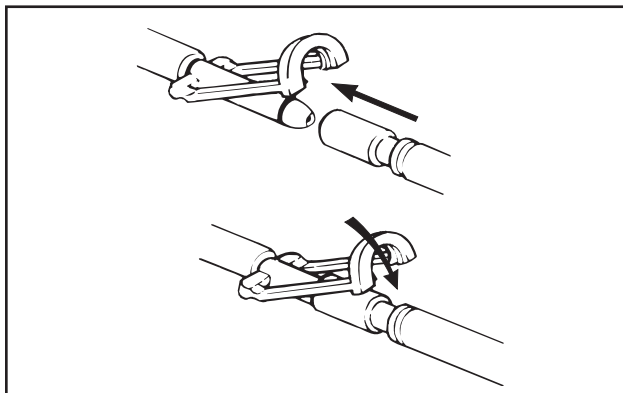
Figure 6—Clik-Tite Connector

3. Two gray hose sets are provided to connect multiple blankets and/or body wraps in order to provide more body surface contact. Increased body surface contact facilitates more efficient warming/cooling.

NOTE: When connecting a second blanket/body wrap to the Medi-Therm III machine, check the water level **prior** to connecting the additional blanket/body wrap (see *Start-up Procedure*, pp. 6-7).

5.4 Open All Clamps

Open all pinch/slide clamps on the connector hose and the Hyper/Hypothermia blanket/body wrap.

Failure to open the pinch/slide clamps will prevent water flow and will trigger the *CHECK FLOW* alert.

5.5 Connect Power/Turn Machine On

Insert the power cord plug into a properly grounded AC receptacle. Press the *ON/OFF* Switch to the *ON* position.

When first turned on, the Medi-Therm III enters its standby mode:

- the *ON/OFF* switch will light;
- the machine will beep once;
- the *WATER*, *PATIENT*, and *SET POINT* temperature displays will show dashes;
- the *SELECT MODE* heading will flash.

5.6 Perform Check Flow Procedure (when using two gray hose sets)

When two gray hose sets are in use, the *CHECK WATER FLOW* alarm will only sound if the flow through both gray hoses is occluded.

To check the flow through the blankets/body wraps:

- Put machine into *BLANKET CONTROL* mode.
- Disconnect one gray hose set from the machine connection. If the flow is patent, the alarm will not sound.
- Reconnect the hose set to the machine.
- Disconnect the second gray hose set from the machine connection. If the flow is patent, the alarm will not sound.
- Reconnect the second hose set to the machine.
- See p.6, section 5.3, part I for instructions on properly connecteing the hoses to the Medi-Therm III.

6.0 Modes of Operation

The Medi-Therm III machine is designed to operate in three (3) modes:

- *BLANKET CONTROL* (water temperature control),
- *PATIENT CONTROL* (automatic patient temperature control), and
- *MONITOR ONLY* (patient temperature monitoring).

When the Medi-Therm III machine is turned on, the set point defaults to 37°C for both the *BLANKET CONTROL* mode and the *PATIENT CONTROL* mode. If either set point is changed, the new value is retained in memory until power is removed.

6.1 BLANKET CONTROL

When operating in *BLANKET CONTROL* mode, the Medi-Therm III machine regulates the water temperature to the selected *SET POINT*. A temperature sensor within the machine monitors water temperature, and the machine heats or cools the water as required to bring it to the *SET POINT* temperature.

In *BLANKET CONTROL* mode, the operator must observe the patient's temperature and manually adjust the water temperature to obtain the desired results.

To operate in *BLANKET CONTROL* mode:

1. Perform all start-up procedures (pp. 6-7).
2. Press the *BLANKET CONTROL* button. The upper window on the top of the *BLANKET CONTROL* button will light.

The *BLANKET* display will indicate water temperature. The *SET POINT* display will indicate the default temperature (37°C) at start-up, until selecting the *SET POINT* temperature in step 3 below.
3. Adjust the *SET POINT* display to the prescribed water temperature setting (4°C to 41°C). See page 5, *SET POINT CHANGE*.

The *STATUS* display will indicate *WARMING* or *COOLING* while the machine is heating or cooling the water. When the water temperature stabilizes (within 1.0°C of the *SET POINT* temperature), both the *WARMING* and *COOLING* lights will turn off indicating that the desired water temperature is being maintained.

NOTE: It takes approximately 8 minutes or less to raise the water temperature from 22°C (room temperature) to 37°C.

NOTE: Within the *SET POINT* range of 30°C to 41°C, if the water temperature does not get to within 1.0°C of the *SET POINT* temperature after 4 hours, the *WATER* display will flash and the audible alarm will sound. Additionally, if the *WATER* temperature reaches the set point and then deviates by more than 1.0°C for more than 10 minutes, the *WATER* display will flash and the audible alarm will sound. When this occurs:

- Pressing the *ALARM SILENCE* button will silence the alarm for 10 minutes.
- Check the patient's condition and consult with a physician.
- Changing the set point will restart the four-hour timer.

⚠ WARNING	
4.	<p>Check the integrity of the skin according to department protocol when regulating temperature with external devices. Frequency of assessment and documentation will vary depending upon the individual response of the patient.</p> <p>Failure to monitor patient may result in skin damage or inappropriate patient temperature.</p>

5. While operating in the *BLANKET CONTROL* mode, the patient probe may be used to monitor the patient's temperature.

To observe patient temperature, insert sensing end of patient probe into patient and tape securely to prevent accidental dislodgment. Insert plug end of patient probe into *PATIENT PROBE* jack. Make sure probe plug is fully seated.

Use only Gaymar probes or equivalent YSI 400 series probes approved for use with medical devices. If in doubt that the correct probe is being used and is operating properly, perform a *PROBE CHECK* test (p. 13).

Patient temperature will be indicated on the *PATIENT* temperature display.

NOTE: In *BLANKET CONTROL* mode, if the machine senses a patient probe temperature below 29°C or above 40°C, the *CHECK PATIENT* alert will light and audible alarm will sound. Therapy will continue. When this occurs:

- Check the patient's temperature.
- Check also to determine if the patient probe has become dislodged, an incorrect probe is being used, or the probe has become defective. See *Troubleshooting* (pp. 12-13).

The *CHECK PATIENT* alert will turn off and the audible alarm will silence when the patient temperature returns to the range of 29°C to 40°C.

6.2 PATIENT CONTROL

When operating in the *PATIENT CONTROL* mode, the Medi-Therm III automatically regulates the patient's temperature to the selected *SET POINT*. The machine constantly compares actual patient temperature with the *SET POINT* value, and automatically adjusts the water temperature so that the desired patient temperature is achieved.

To operate in *PATIENT CONTROL* mode:

1. Perform all start-up procedures (pp. 6-7).
2. Insert sensing end of patient probe into patient and tape securely to prevent accidental dislodgment.
3. Insert plug end of patient probe into *PATIENT PROBE* jack. Make sure probe plug is fully seated.
Use only Gaymar probes or equivalent YSI 400 series probes approved for use with medical devices. If in doubt that the correct probe is being used and is operating properly, perform a *PROBE CHECK* test (p. 13).
4. Press the *PATIENT CONTROL* button. The upper window on the top of the *PATIENT CONTROL* button will light.

NOTE: To prevent accidental triggering of the *CHECK PATIENT* alert, place probe in patient and wait for patient temperature to stabilize before pressing *PATIENT CONTROL* button.

The *WATER* display will indicate water temperature. The *PATIENT* display will indicate patient temperature. The *SET POINT* temperature will indicate the default temperature (37°C)

at start-up, until selecting the *SET POINT* temperature in step 5.

5. Adjust the *SET POINT* display to the prescribed patient temperature setting (30°C to 39°C). See page 5, *SET POINT CHANGE*. The *STATUS* display will indicate *WARMING* or *COOLING* while the machine is heating or cooling the patient. When the patient temperature stabilizes (within 0.5°C of the *SET POINT* temperature) both the *WARMING* and *COOLING* lights will turn off indicating that the desired patient temperature is being maintained.
6. The rate of patient cooling and warming is user adjustable through the use of the *CONTROL OPTION*. The *CONTROL OPTION* defaults to *RAPID* when *PATIENT CONTROL* mode is entered. Select a different *CONTROL OPTION* by pressing the *PATIENT CONTROL* button again.

When COOLING the patient:

RAPID - the coldest water is used for cooling - as low as 4°C. This may not be the most comfortable setting for the patient.

MODERATE - water temperature is limited to 15°C below the patient's temperature. This will provide an improved comfort setting to the patient.

GRADUAL - water temperature is limited to 10°C below the patient's temperature. This will provide an improved comfort setting to the patient.

When WARMING the patient*:

RAPID - the highest allowable water temperature is used for warming - as high as 41°C. This is the fastest warming mode the machine can provide.

MODERATE - the patient's temperature will increase at a rate of 1°C in a period of 3 hours (0.33°C/Hr).

GRADUAL - the patient's temperature will increase at a rate of 1°C in a period of 6 hours (0.17°C/Hr).

When using *AUTO MODERATE* and *AUTO GRADUAL* for patient warming, **NO MANUAL INTERVENTION** is required once the patient set point temperature has been selected for warming.

*NOTE: The *PATIENT* temperature display will begin to flash, the *ALERT* LED will flash and the audible alarm will toggle when the patient's temperature deviates by 1.0°C or more from the desired warming rate. This alarm only applies when **warming** a patient in **MODERATE** or **GRADUAL** modes.

⚠ WARNING

In *PATIENT CONTROL* mode, if the machine senses a patient probe temperature below 29°C or above 45°C, therapy will stop. The *CHECK PATIENT* alert and audible alarm will be on. When this occurs:

- **Check the patient's temperature.**
- **Check also to determine if the patient probe has become dislodged, an incorrect probe is being used, or the probe has become defective (see *Troubleshooting*, pp. 12-13).**
- **Switch to *BLANKET CONTROL* mode if continued therapy is desired.**

NOTE: For patient probe temperatures between 40°C and 45°C, therapy will still be provided even though the *CHECK PATIENT* alert light and audible alarm are on.

NOTE: If the *PATIENT* temperature does not get to within 0.5°C of the *SET POINT* temperature after 4 hours, the *PATIENT* display will flash and audible alarm will sound.

When this occurs:

- Pressing the *ALARM SILENCE* button will silence the alarm for 10 minutes.
- Check the patient's condition and consult with a physician.
- Switch to *BLANKET CONTROL* mode if continued therapy is desired.
- Changing the set point will restart the four-hour timer.

NOTE: When using the rate controlled *AUTO MODERATE* or *AUTO GRADUAL* modes for warming; switching to other modes, e.g. *AUTO RAPID* or *MANUAL*, or changing the temperature set point during the therapy will cause the Medi-Therm to reset the therapy. This may impact the overall duration of the therapy depending on when in the hourly cycle the mode is interrupted. The therapy time duration is also dependent on the patient's temperature at the time the *AUTO MODERATE* or *AUTO GRADUAL* modes are re-selected.

6.3 MONITOR ONLY (Patient Temperature Monitoring, No Therapy)

Press *MONITOR ONLY* when no therapy is required, but monitoring of patient temperature is desired.

To operate in *MONITOR ONLY* mode:

1. Insert sensing end of patient probe into patient and tape securely to prevent accidental dislodgment.
2. Insert plug end of patient probe into the *PATIENT PROBE* jack. Make sure probe plug is fully seated.

Use only Gaymar probes or equivalent YSI 400 series probes approved for use with medical devices. If in doubt that the correct probe is being used and is operating properly, perform a *PROBE CHECK* test (p. 13).

3. Press the *MONITOR ONLY* button. The upper window on the top of the *MONITOR ONLY* button will light. The patient's temperature will be displayed.

If the machine senses a patient probe temperature below 29°C or above 40°C, the *CHECK PATIENT* alert will light and an audible alarm will sound.

REMEMBER—
in *MONITOR ONLY*, no therapy is provided.

7.0 Shutdown Procedure

When therapy is completed, turn off the Medi-Therm III machine. Close the pinch clamps before disconnecting the blanket(s)/body wrap(s).

NOTE: Do not discard the DBK35CE Connector Hose.

8.0 Safety Systems

8.1 Overtemp Safety Circuits

In the event of a controller failure, there are two separate backup circuits which prevent the water temperature from surpassing safe operating limits.

The first overtemp backup consists of an electronic sensing circuit, totally independent of the main control circuitry. This circuit will interrupt heater power and initiate a *REMOVE FROM USE NOW* audible and visual alert to keep the water temperature below 42°C. The second overtemp backup is a bimetallic thermostat which will directly interrupt power to the heater in the event that both the main control circuitry and the first backup fail. This device will also initiate a *REMOVE FROM USE NOW* alert.

8.1.1 Overtemp Verification Test

Perform the following test to verify operation of the first overtemp safety backup without having to open up the machine. Refer to the *Medi-Therm III MTA7912 Service Manual* if complete service testing of the machine is required.

1. Disconnect the hoses from the *SUPPLY* and *RETURN* fittings on the front of the machine.
2. Turn the machine on while simultaneously pressing the *TEST LIGHTS* button. This will put the unit into a *TEST MODE*. In this *TEST MODE*, the water temperature display will light, the set point will display "0", and the temperature will start to increase.

NOTE: If the unit flashes *CHECK FLOW*, the hoses have not been disconnected and the temperature will not increase. Disconnect the hoses.

3. If the first overtemp safety circuit is working properly, the unit should go into a *REMOVE FROM USE NOW* alert condition. The audible indicator will remain on continuously. The entire test should take approximately 3 minutes.

If the unit goes into a *REMOVE FROM USE NOW* alert condition, but the audible indicator does not remain on, the first overtemp backup is not functioning and the second overtemp backup has tripped. Do not use the machine. Request service.

If the unit does not go into a *REMOVE FROM USE NOW* alert condition, do not use the machine. Request service.

4. Turn the machine off. This will end the *TEST MODE*.
5. Reconnect hoses to the fittings on the front of the unit.
6. Turn the unit on and select *BLANKET CONTROL* mode immediately. This will allow water to flow past the first over-temp probe and reset it.

NOTE: If the machine goes into a *REMOVE FROM USE NOW* alert condition again when selecting *BLANKET CONTROL*, the second over-temp backup device (the thermostat) has also tripped. Turn the machine off. Connect a hose to the *SUPPLY* fitting. Direct the other end of the hose into a water container so that water from the cold reservoir will siphon past the thermostat and reset it. Allow water to flow for approximately two minutes.

9.0 Troubleshooting

CHECK FLOW	Check for hoses fully inserted, one into return (top) and one into supply (bottom).
	Check to be sure all pinch clamps are open.
	Check for kinked hoses or creased blanket/body wrap.
CHECK PATIENT	Make certain patient probe is fully inserted into the patient probe jack.
	Check for secure connection to patient. (If probe is sensing ambient, alarm will sound.)
	Check for proper type of probe (Gaymar probe or equivalent YSI 400 series probe approved for use with medical devices) and adaptor, if needed.
ADD WATER	Check to make sure water is at the proper level. If the green band on the float is not visible, fill machine with distilled water until the green band is visible.
REMOVE FROM USE NOW	Indicates that the machine has shut down due to the water temperature exceeding the high or low limit. Or, a system error has been detected. Remove the machine from use immediately and notify medical equipment repair personnel.
POWER LOSS	Check to make certain power cord is fully engaged into properly grounded receptacle.
	Make certain the receptacle has power.
	Reset unit circuit breaker by turning OFF then ON. If circuit breaker trips again, refer to medical equipment repair personnel.
BLANKET/BODY WRAP WILL NOT HEAT	Check for water circulating through the blanket/body wrap. See CHECK FLOW. Do not fold blankets/body wraps.
	Check to make sure the temperature controller is working by adjusting the setpoint (while in BLANKET CONTROL mode) at least 2°C above the water temperature. This should cause the WARMING indicator to light.
BLANKET/BODY WRAP WILL NOT COOL	Check for water circulating through the blanket/body wrap. See CHECK FLOW. Do not fold blankets/body wraps.
	Check to make sure the rear and side vents are not blocked. Inlet and exhaust grills should be kept clear and clean to allow air to circulate freely.
	Check to make sure the temperature controller is working by adjusting the setpoint (while in BLANKET CONTROL mode) at least 2°C below the water temperature. This should cause the COOLING indicator to light.

9.1 Probe Check

⚠ WARNING

When performing the *PROBE CHECK*, use a protective sheath on the probe. (Becton-Dickinson catalog 3700 oral sheath or equivalent.)

Failure to use a sheath could result in cross-contamination.

1. Insert probe into protective sheath and then into *PROBE CHECK WELL* on front of machine base (fig. 7).
2. Operate machine in *BLANKET CONTROL* mode with a blanket/body wrap connected. Adjust *SET POINT* temperature to between 33°C and 37°C. Wait until machine temperature stabilizes (both the *WARMING* and *COOLING* lights are off and patient temperature display stabilizes). Water and patient temperature displays should read within 2°C of each other. If not, patient probe may be defective or incorrect probe is being used.

Use only Gaymar probes or equivalent YSI 400 series probes approved for use with medical devices.

3. If probe appears defective, try another probe and repeat *PROBE CHECK*. If a new, correct probe does not meet this test, remove machine from use and request service.

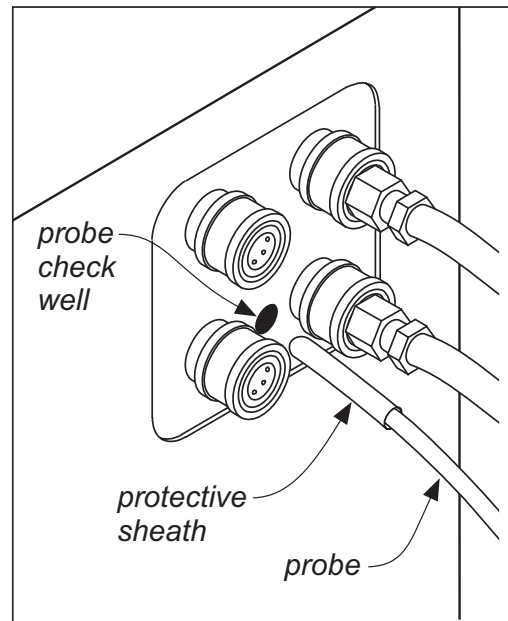
Figure 7—Probe Check Well

10.0 Care and Cleaning

As with any precision equipment, proper care and maintenance will insure proper performance and reliability. The following routine maintenance procedures will help insure dependable Medi-Therm III operation.

10.1 Machine Care

▲ CAUTION

- Fill with distilled water only.
Failure to use distilled water may result in poor machine performance.
- **Do not** use alcohol. Alcohol may accelerate blanket/body wrap deterioration. Alcohol is flammable.
- **Do not** operate without water, since damage to internal components may result.
- **Do not** overfill. Overfilling may result in overflow because the water in the blanket/body wrap drains back into the machine when the machine is turned off.
- When filling an empty Medi-Therm III machine, add a small amount of germicidal or sanitizer solution to the water (Air-Kem A-33 liquid or equivalent). Carefully follow the manufacturer's recommended concentration to prevent algae growth within the Medi-Therm III system and Hyper/Hypothermia blanket/body wrap.
Do not use too much concentrate, since it will result in "foaming," which can reduce the blanket/body wrap efficiency and may damage the circulating pump.
- **Do not** use bleach. Bleach will damage the heating element.

10.2 Draining the Machine

It is not necessary to drain the Medi-Therm III machine after each use. However, periodic draining is recommended at least once a month.

10.3 Machine Cleaning

▲ CAUTION

Users should not use cleaning or decontamination methods different from those recommended.

To clean the external surfaces of the machine and connector hose, use a nonabrasive cleaning solution (such as warm, soapy water) and a clean cloth. Wipe or air dry. Apply a disinfectant such as 10% chlorinated bleach solution (chlorinated bleach with 5.25% sodium hypochlorite) to the external surfaces and allow to dry.

10.4 Blanket/Body Wrap Care

▲ CAUTION

- Do not store filled blankets/body wraps. Algae growth may occur inside.
- Do not use a blanket/body wrap if severe algae buildup occurs.
- Do not expose blankets/body wraps to temperatures over 65°C.

Disposable Blankets/Body Wraps

1. After the machine has been turned off, raise the blanket/body wrap higher than the machine to allow most of the water to drain back into the machine.
2. Close both sets of hose clamps.
3. Disconnect the blanket/body wrap from the machine.
4. Dispose of blanket/body wrap.

Reusable Blankets

1. After the machine has been turned off, raise the blanket higher than the machine to allow most of the water to drain back into the machine.
2. Close both sets of hose clamps.
3. Disconnect the blanket from the machine.
4. Manually clean both sides of the blanket on a flat surface with warm water, a mild commercial detergent, and a sponge or cloth.

5. Thoroughly rinse with clean water for 30 seconds.
6. Air dry or wipe with a clean cloth.
7. Apply a disinfectant such as a 10% chlorinated bleach solution (chlorinated bleach with 5.25% sodium hypochlorite) to both sides of the blanket.
8. Allow to air dry. Solution contact time is what makes disinfection effective.

NOTE: Excess solution pooled in the buttons can be removed with a clean, dry cloth.

9. Loosely roll up the blanket and store for next use.
Reusable blankets can be cleaned and reused so long as they do not crack or leak.

10.5 Probe Care

Reusable Probes

For cleaning, disinfecting, and sterilizing reusable Gaymar probes (or equivalent YSI 400 series reusable probes), refer to the instructions provided with the probe.

Disposable Probes

Gaymar disposable probes should be discarded after use.

10.6 Probe Check Well Care

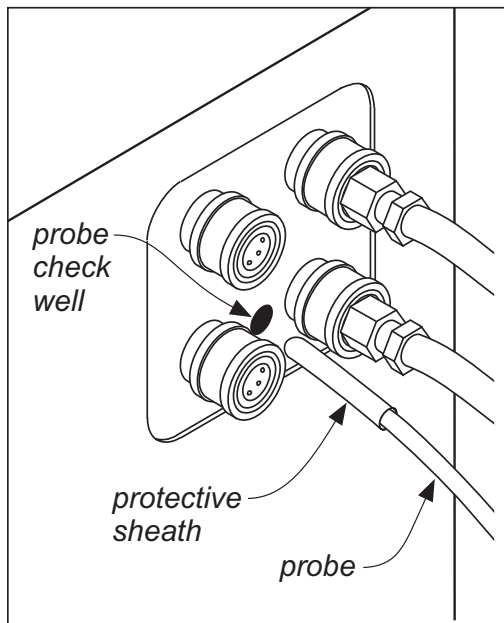
Figure 8—Probe Check Well

The *PROBE CHECK WELL* should be cleaned with a small tubular brush as outlined in section 10.3, *Machine Cleaning*.

10.7 Quick-Disconnects

The quick-disconnects joining the connector hose to the machine may become difficult to engage. This problem can be prevented by periodically applying a silicone base or light machine oil to the outside of the male connector prior to engagement.

10.8 Preventive Maintenance and Service

Preventive maintenance and service procedures are described in the *Medi-Therm III MTA7912 Service Manual*. These procedures should be performed annually (or as required by hospital protocol) by qualified personnel such as certified biomedical electronics technicians or certified clinical engineers.

11.0 Blankets/Body Wraps/Accessories

Single Patient Use Blankets, Polymer with Nonwoven Fabric

- REF DHP810 O.R. Table and General Use (64 cm x 163 cm)
- REF DHP812 Pediatric General Use (56 cm x 84 cm)
- REF DHP813 Adult General Use (76 cm x 163 cm)

Single Patient Use Blankets, All Vinyl

- REF DHP901 Adult General Use (64 cm x 175 cm)
- REF DHP902 Pediatric General Use (64 cm x 91 cm)

Reusable Blankets, All Vinyl

- REF HP7010 Adult General Use (64 cm x 175 cm)
- REF HP7020 Pediatric General Use (64 cm x 91 cm)

Single Patient Use Torso Wraps, Polymer with Nonwoven Fabric

- REF DHV530 Small/Medium
- REF DHV535 Large

Single Patient Use Leg Wraps, Polymer with Nonwoven Fabric

- REF DHL540 One size fits all

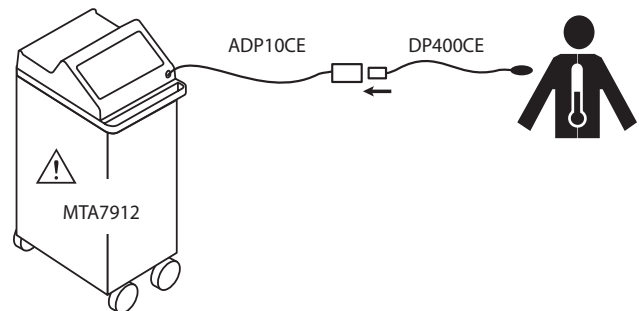
All blankets/body wraps require use of the reusable DBK35CE Connector Hose.

Accessories *

- REF DBK35CE Insulated Connector Hose (3 meters)
- REF PAT101CE Reusable Adult Patient Probe (3 meters) Rectal/Esophageal
- REF PAT102CE Reusable Pediatric Patient Probe (3 meters) Rectal/Esophageal
- REF PAT108CE Reusable Patient Probe (3 meters) Skin Surface

- REF DP400CE Disposable Adult and Pediatric Patient Probe (0.9 meters) Rectal/Esophageal [requires ADP10CE Adaptor Cable]
- REF ADP10CE Reusable Probe Adaptor Cable for DP400CE Probe

* These Gaymar probes, adapters, and hoses should be used only with Gaymar Hyper/Hypothermia Machines.

12.0 Customer Training

In-service training is recommended and can be arranged through your Gaymar dealer.

13.0 Warranty

The Medi-Therm III Hyper/Hypothermia machine is warranted free of defects in material and workmanship for a period of two years under the terms and conditions of the Gaymar warranty in place at the time of purchase. The compressor portion of the machine carries a five year prorated warranty. Contact your local dealer if you have questions about warranty or repair.

13.1 Return Authorization

Please contact your local dealer to obtain an RG (Returned Goods) number prior to returning the machine.

NOTE: Machines will not be accepted for repair unless they have been cleaned and disinfected.

14.0 Specifications

Physical

Dimensions	94.0 cm (height) x 35.6 cm (width) x 47.6 cm (depth)
Floor area	0.17 m ²
Weight	66.2 kg empty; 73.9 kg full
Reservoir capacity	9.5 L distilled water
Refrigerant	HFC-134a (does not contain chlorine)
Power loss indicator battery	8.4 V NiMH
Power cord	Detachable, 4.6 meters. Use only an international (harmonized) 3-wire cord using cordage approved to HD-21 conductor size 1.00mm ² (HO5VVVF3GI.00).

Performance

Patient temperature measurement accuracy:	
with Gaymar disposable probe	±0.4°C (±0.2°C display accuracy; ±0.2°C probe accuracy)
with Gaymar reusable probe	±0.3°C (±0.2°C display accuracy; ±0.1°C probe accuracy)
Heater	500 W at 240 V
Flow rate with Gaymar blanket	1 liter per minute
Ambient operating temperature	15.6°C to 32.2°C
Storage temperature	>0°C

Electrical

Voltage/frequency	~220 V to ~240 V ±10%, 50 Hz
Current	6 A
Circuit breaker	10 A

Regulatory

Medical Device Directive (93/42/EEC) Class II

Classification

EN 60601-1 Class I, grounded, not suitable for use in the presence of flammable anaesthetic mixture with air or with oxygen or nitrous oxide. Not classified for protection against harmful ingress of liquid.

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Gaymar Industries, Inc.