

1020 Full-Length X-Ray Stretcher
1020 Du chariot d'urgence sur tout la longueur
1020 Röntgen und Notfallliege voller Länge
1020 Brancard voor Lengte-as Röntgenopnamen
1020 Barella per Radiografie su tutta l'asse longitudinale
Camilla 1020 con rayos X toda su longitud
Maca de Raio-X a Todo o Comprimento 1020
1020 Fuldt Udtrukken Røntgenbår
1020 Fullt Utdragit Røntgenbår

Parts Manual

For replacement parts or service, contact your Stryker representative.

Pour les pièces de rechange vous pouvez vous adresser à votre fournisseur de Stryker.

Für Ersatzteile können Sie sich an Ihren Stryker-Lieferanten wenden.

Voor reserveonderdelen kunt u zich wenden: tot uw Stryker leverancier.

Per i pezzi di ricambio potete rivolgervi al Vostro fornitore di prodotti Stryker.


Para piezas de recambio Uds. pueden dirigirse a su proveedor de Stryker

Para obtenção de peças sobressalentes favor dirigir-se ao seu vendedor Stryker.

Henvend Dem venligst til Deres Stryker forhandler for reservedele eller teknisk assistance.


Reservdelar och teknisk rådgivning får du av din närmaste STRYKER-distributör.

0753-006-110

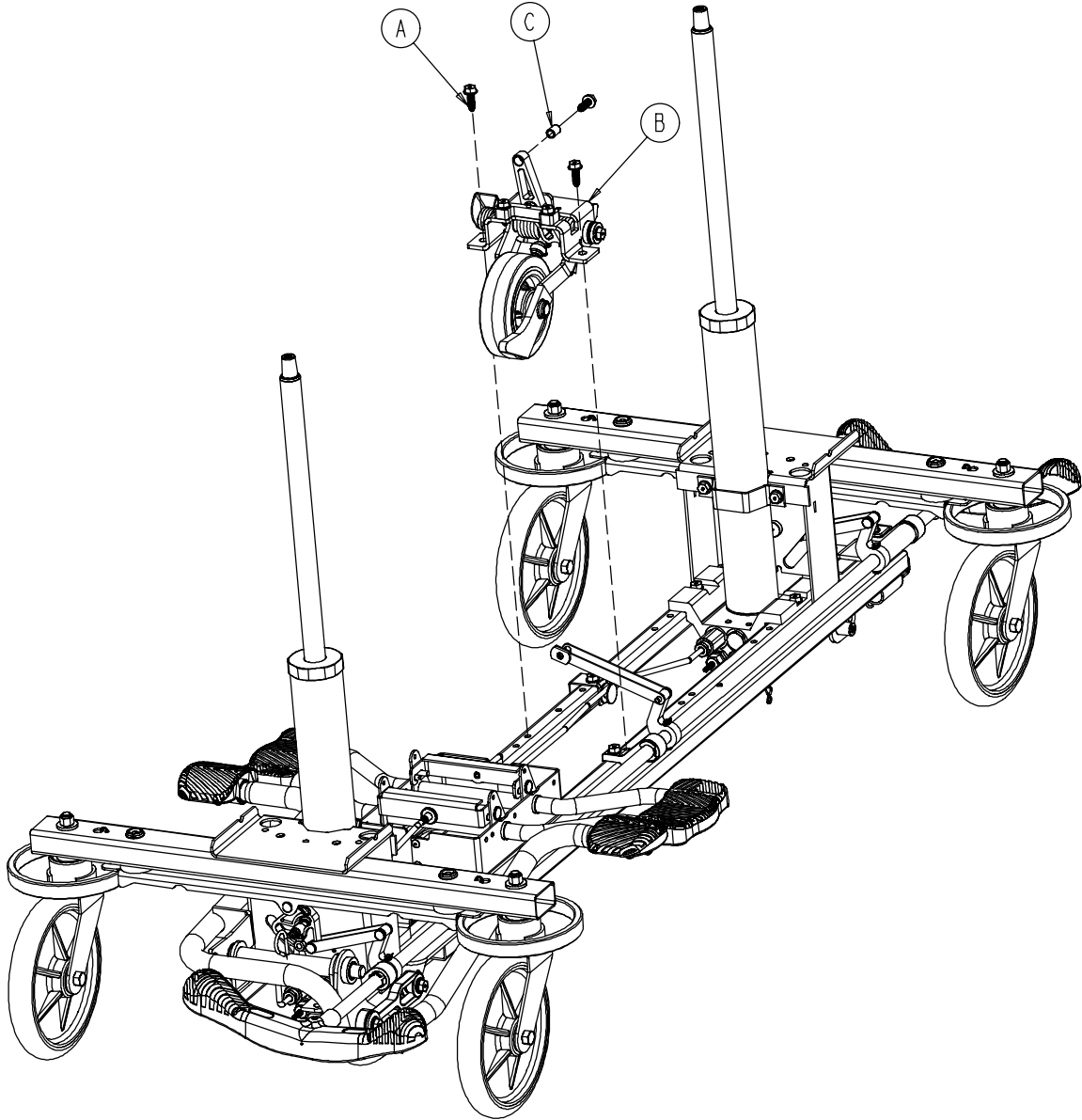


A	0023-288-000	3
B	0753-006-130	1
C	0753-006-148	1

0753-006-130

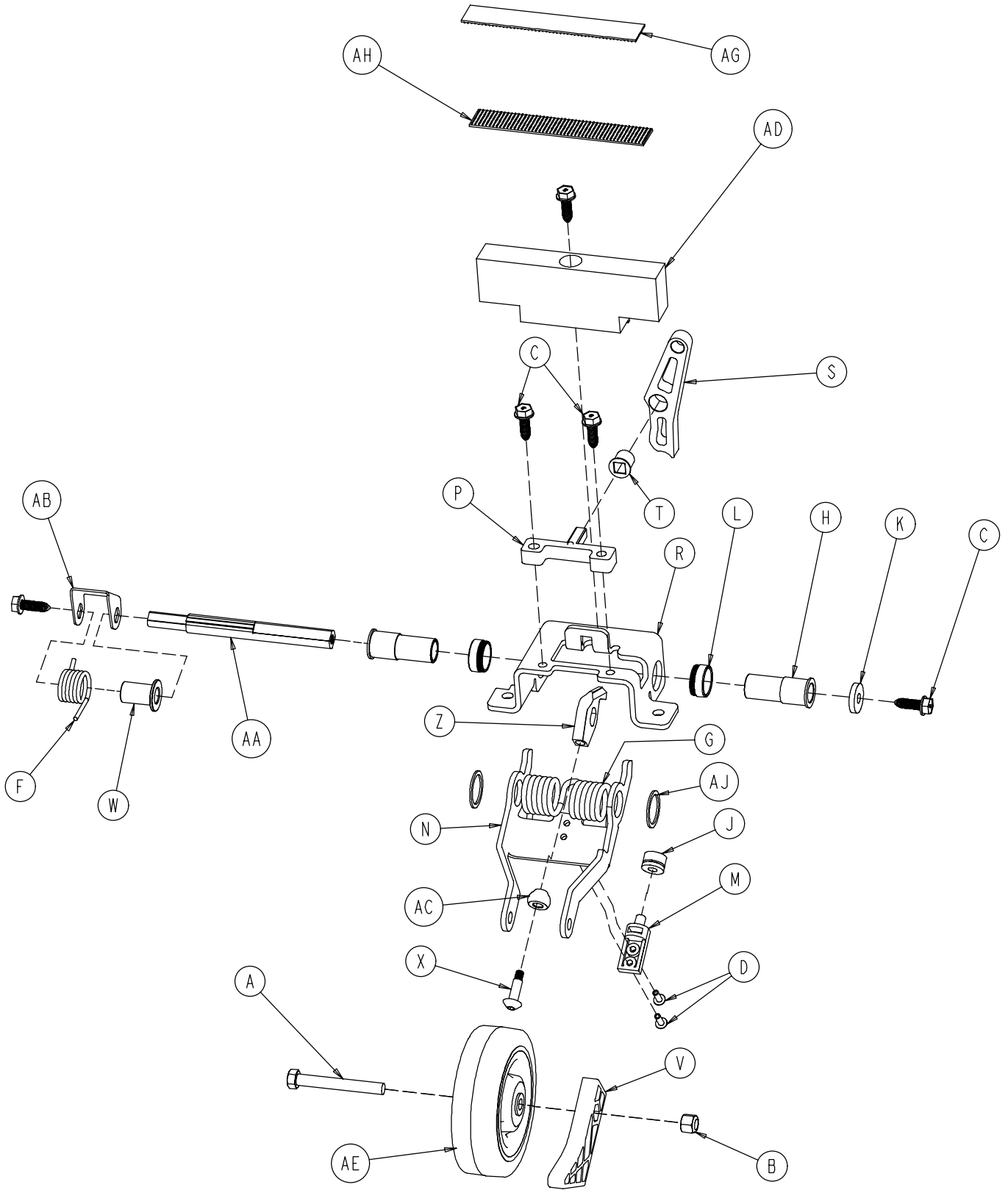

0753-006-130


A	0003-083-000	1
B	0016-035-000	1
C	0023-288-000	5
D	0025-050-000	2
E	0038-525-000	1
F	0753-006-074	1
G	0753-006-075	1
H	0753-006-097	2
J	0753-006-106	1
K	0753-006-108	1
L	0753-006-115	2
M	0753-006-120	1
N	0753-006-126	1
P	0753-006-133	1
R	0753-006-134	1
S	0753-006-142	1
T	0753-006-143	1
V	0753-006-149	1
W	0753-006-152	1
X	0753-006-153	1
Z	0753-006-198	1
AA	0753-006-223	1
AB	0753-006-227	1
AC	0753-006-277	1
AD	0753-010-045	1
AE	1210-001-147	1
AG	0753-006-280	1
AH	0753-006-281	1
AJ	0011-360-000	2

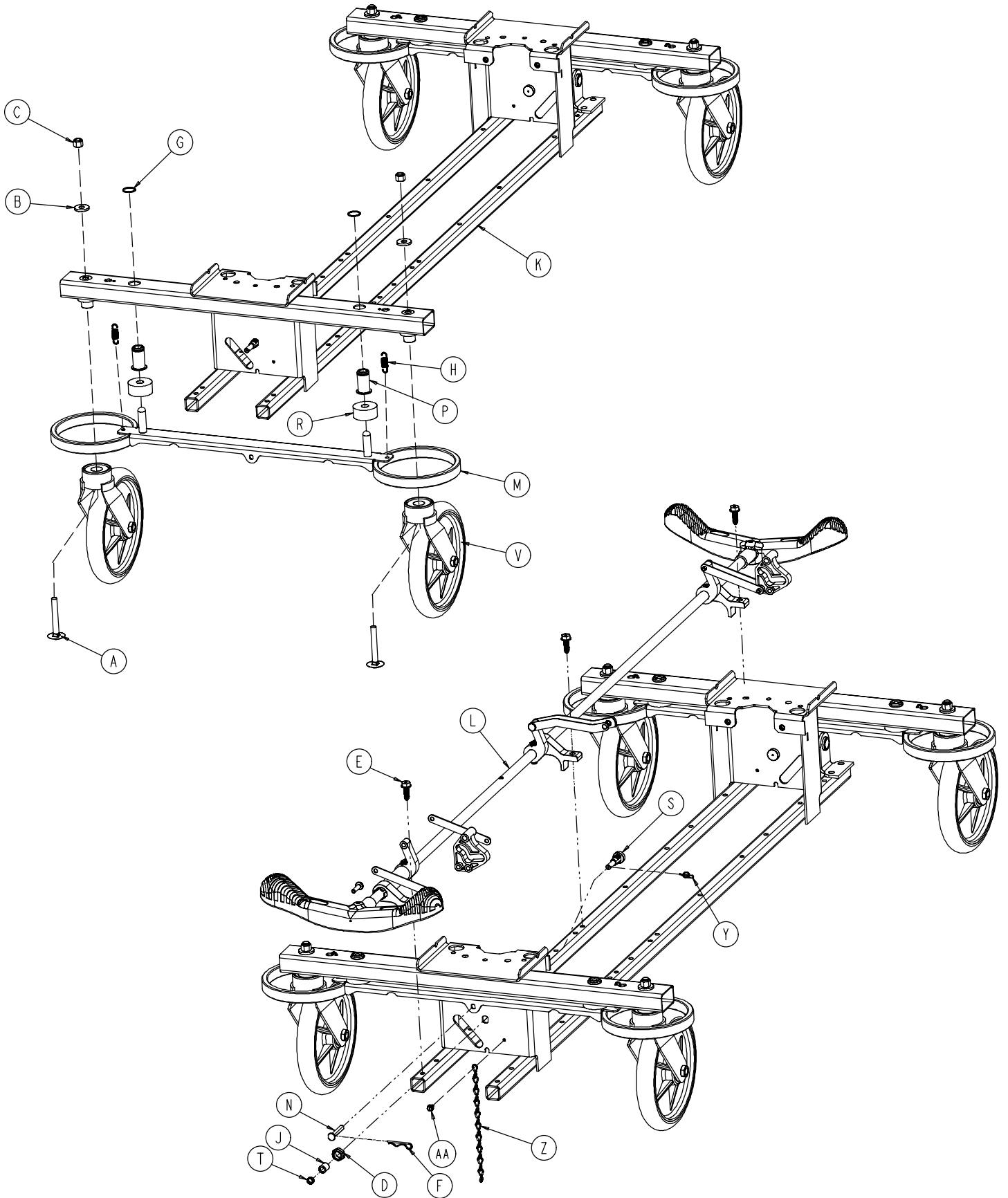

0753-003-005


A	0005-039-000	4
B	0011-262-000	4
C	0016-035-000	4
D	0016-049-000	2
E	0023-288-000	3
F	0027-012-000	2
G	0028-037-000	4
H	0038-439-000	4
J	0081-272-000	2
K	0753-001-001	1
L	0753-003-001	1
M	0753-003-006	2
N	0753-003-066	2
P	0753-003-079	4
R	0753-003-121	4
S	0753-003-130	2
T	0753-003-131	2
V	0753-010-020	4
Y	0027-019-000	2
Z	0753-010-012	1
AA	0023-025-000	1

0753-003-001

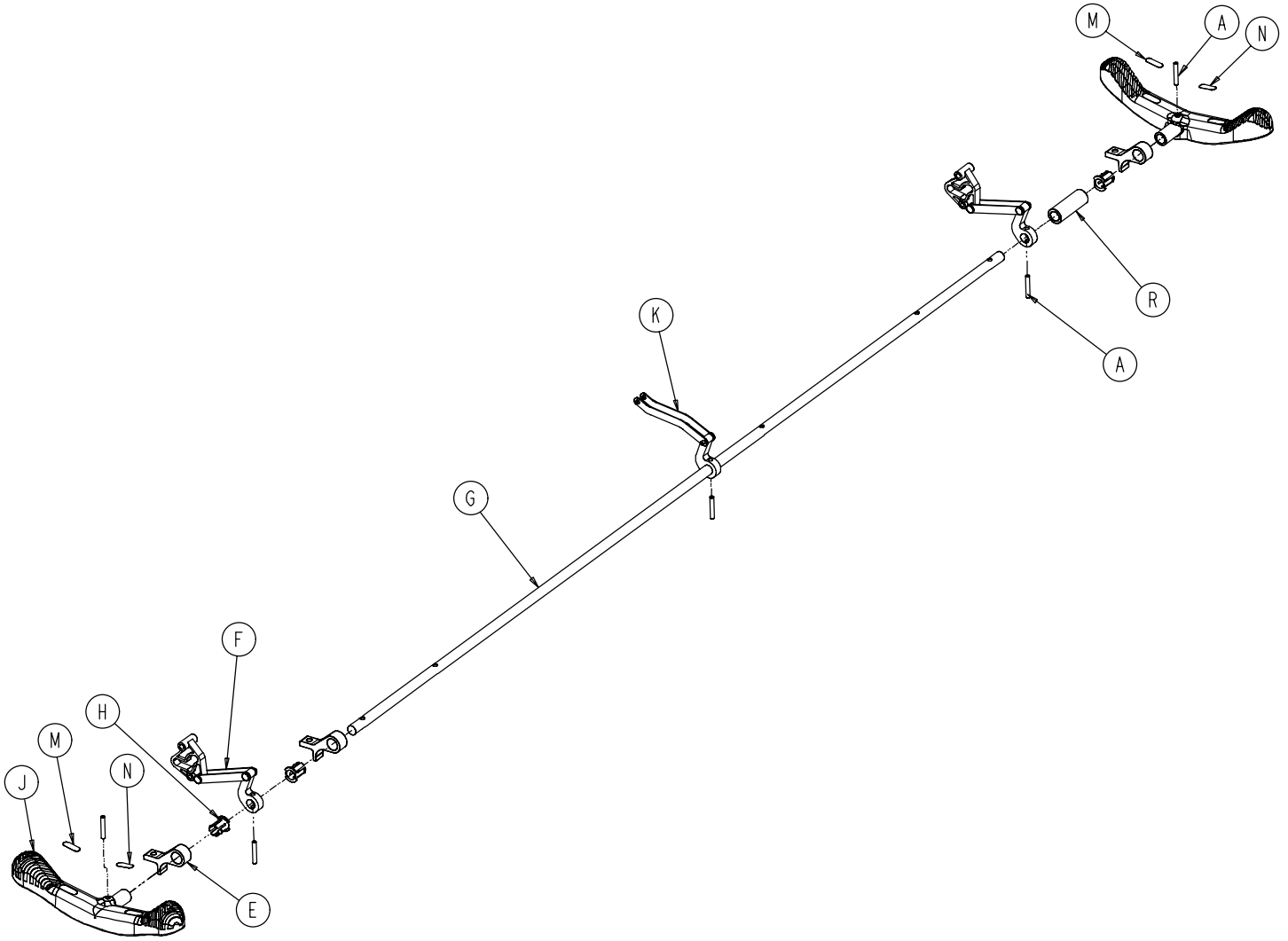


A	0026-067-000	2
E	0753-003-004	3
F	0753-003-010	2
G	0753-003-014	1
H	0753-003-015	3
J	0753-003-099	2
K	0753-006-135	1
M	1210-201-335	2
N	1210-201-336	2
R	0753-003-133	1

0753-003-010

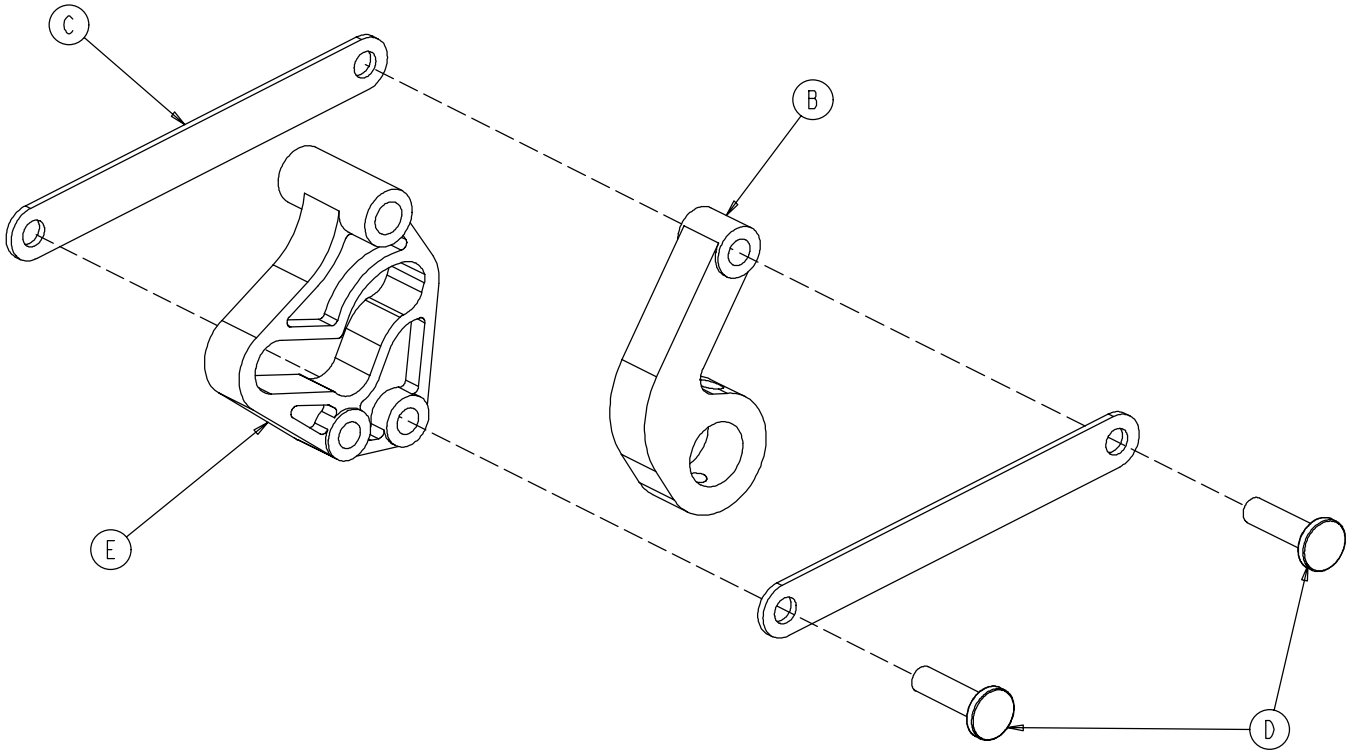


B	0753-003-011	1
C	0753-003-061	2
D	0753-003-098	2
E	0753-003-102	1

0753-006-135

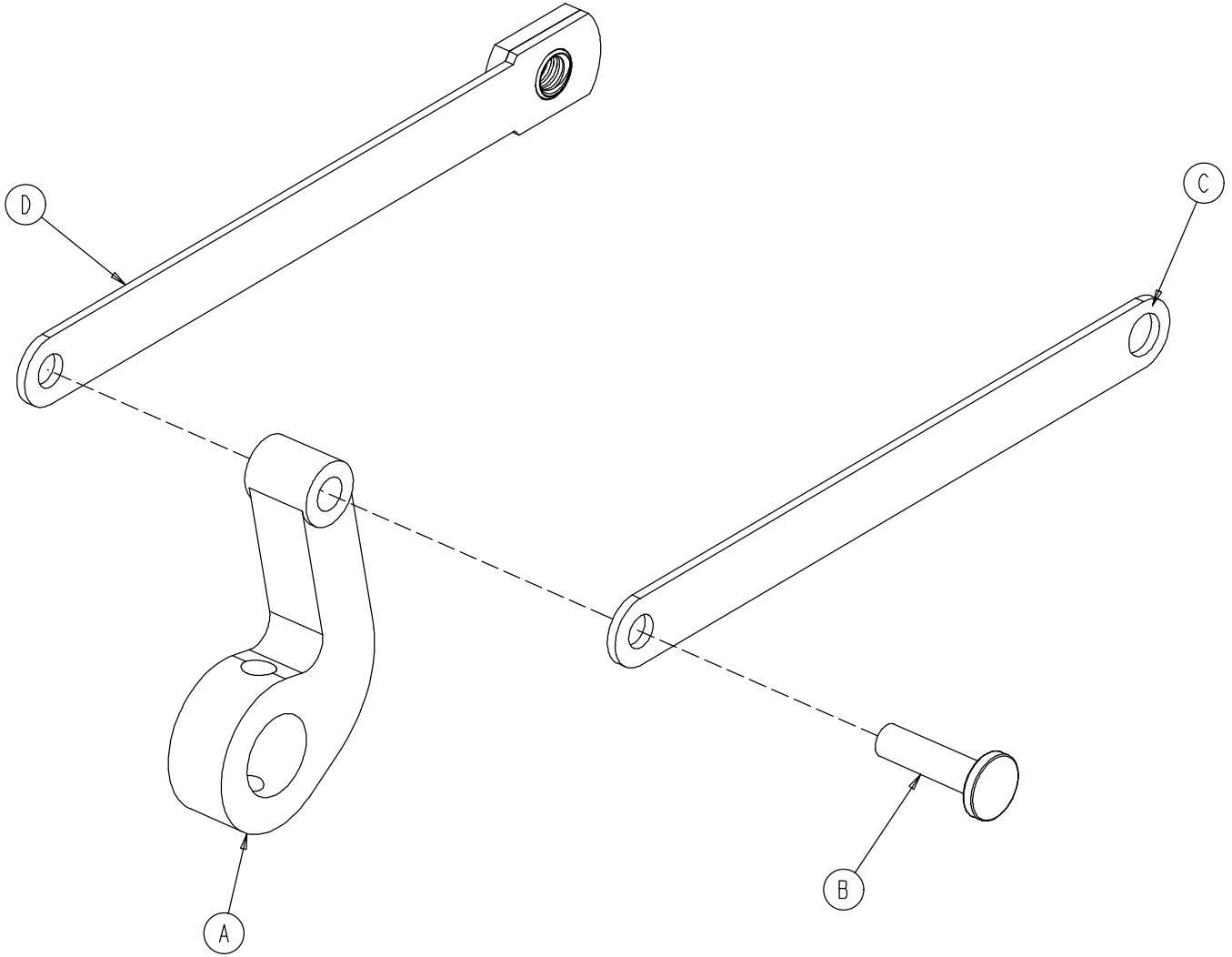


A	0753-003-011	1
B	0753-003-098	1
C	0753-006-122	1
D	0753-006-147	1


0753-010-020

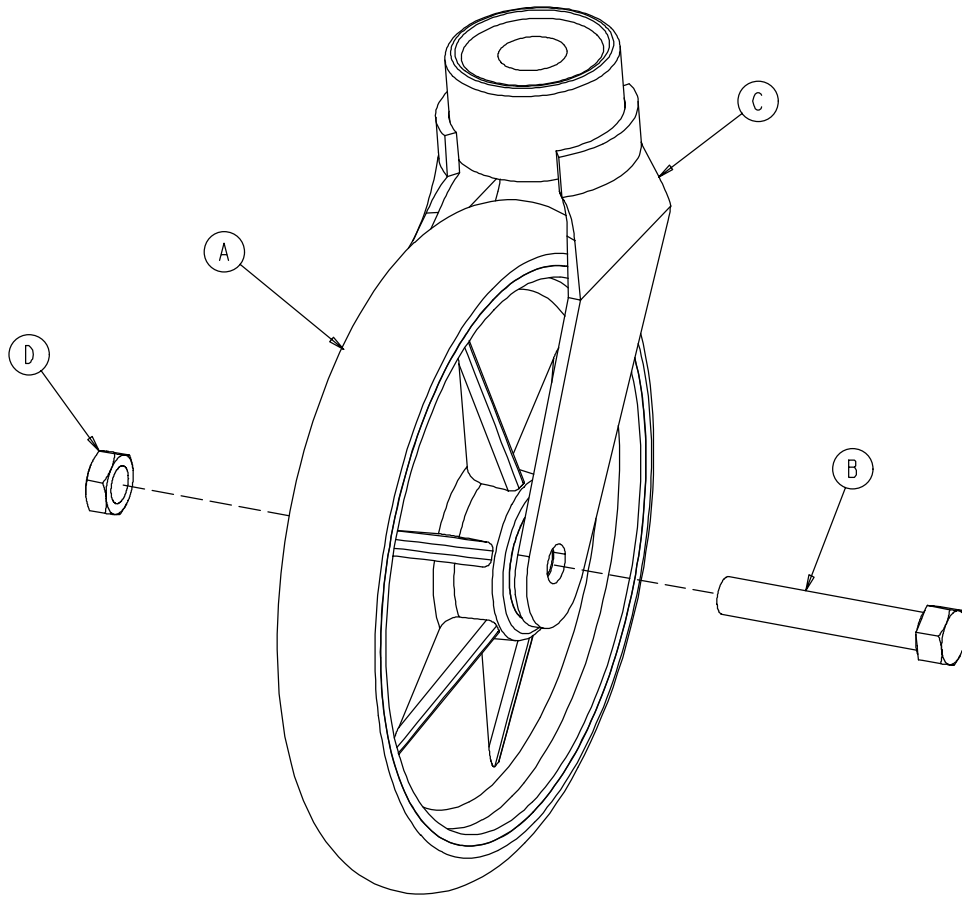

A	0715-002-025	1
B	0003-099-000	1
C	0753-010-021	1
D	0016-060-000	1

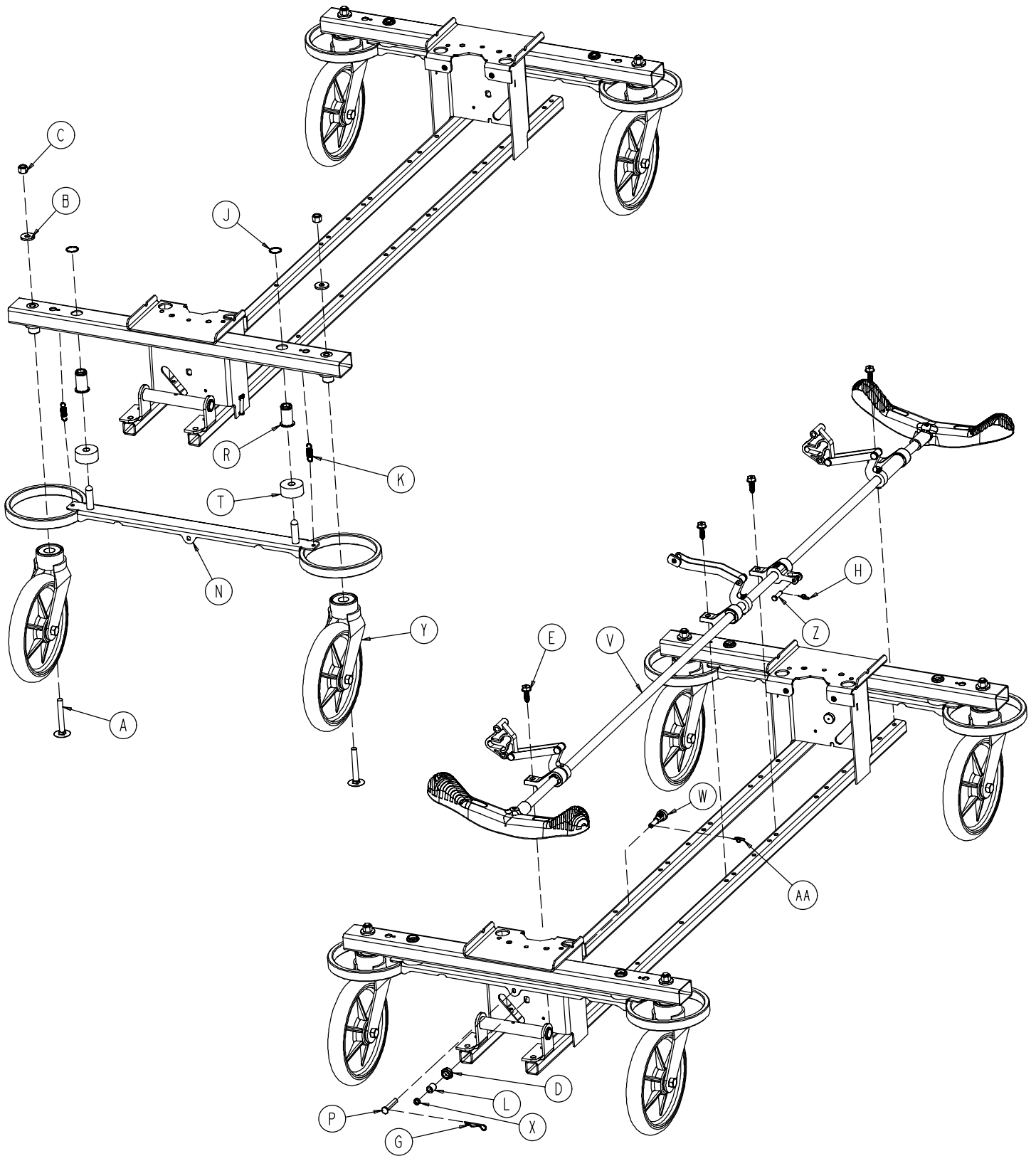


0753-003-120

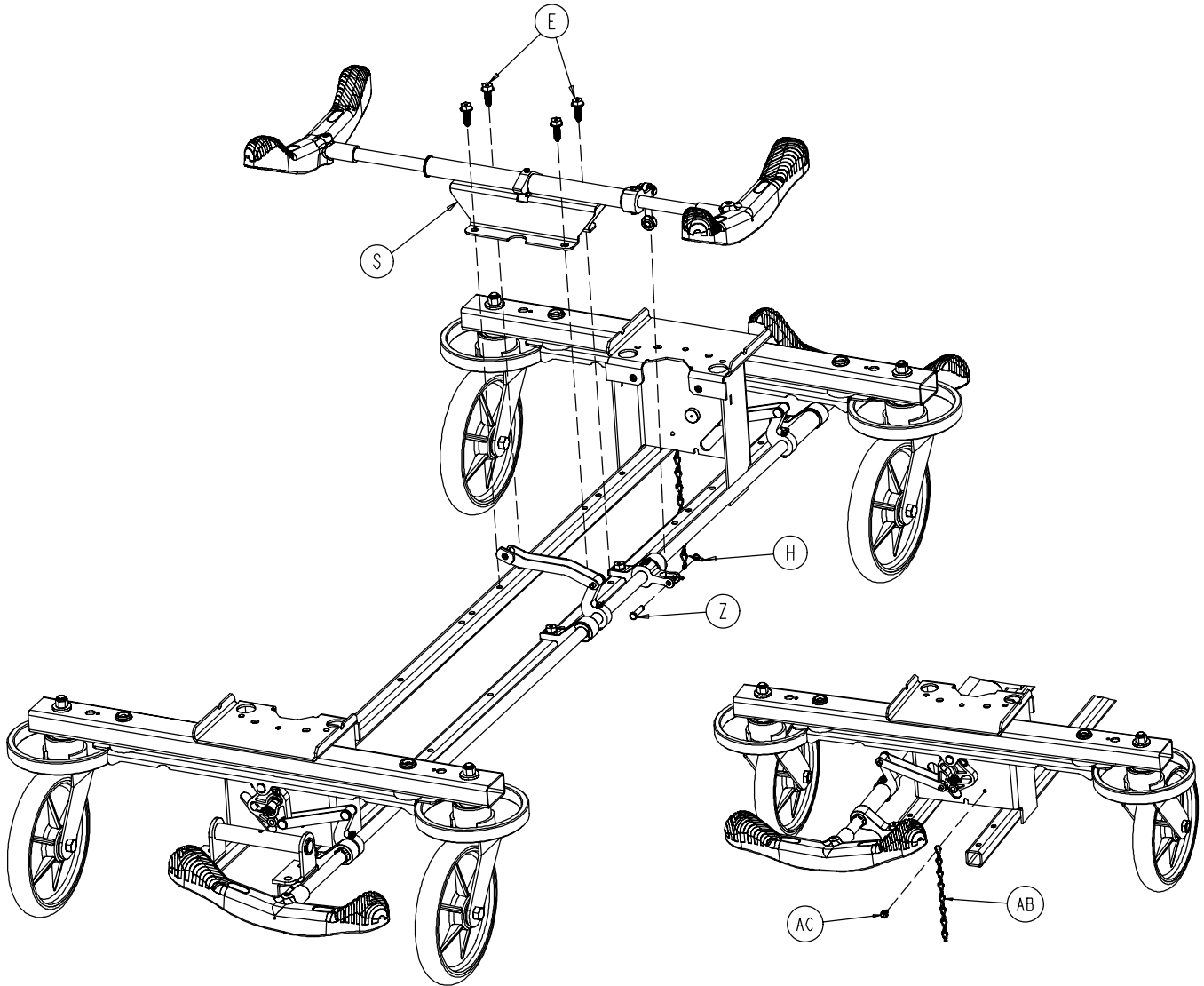


A	0005-039-000	4	P	0753-003-066	2
B	0011-262-000	4	R	0753-003-079	4
C	0016-035-000	4	S	0753-003-110	1
D	0016-049-000	2	T	0753-003-121	4
E	0023-288-000	8	V	0753-003-125	1
G	0027-012-000	2	W	0753-003-130	2
H	0027-020-000	1	X	0753-003-131	2
J	0028-037-000	4	Y	0753-010-020	4
K	0038-439-000	4	Z	0026-340-000	1
L	0081-272-000	2	AA	0027-019-000	2
M	0753-001-001	1	AB	0753-010-012	1
N	0753-003-006	2	AC	0023-025-000	1

0753-003-110

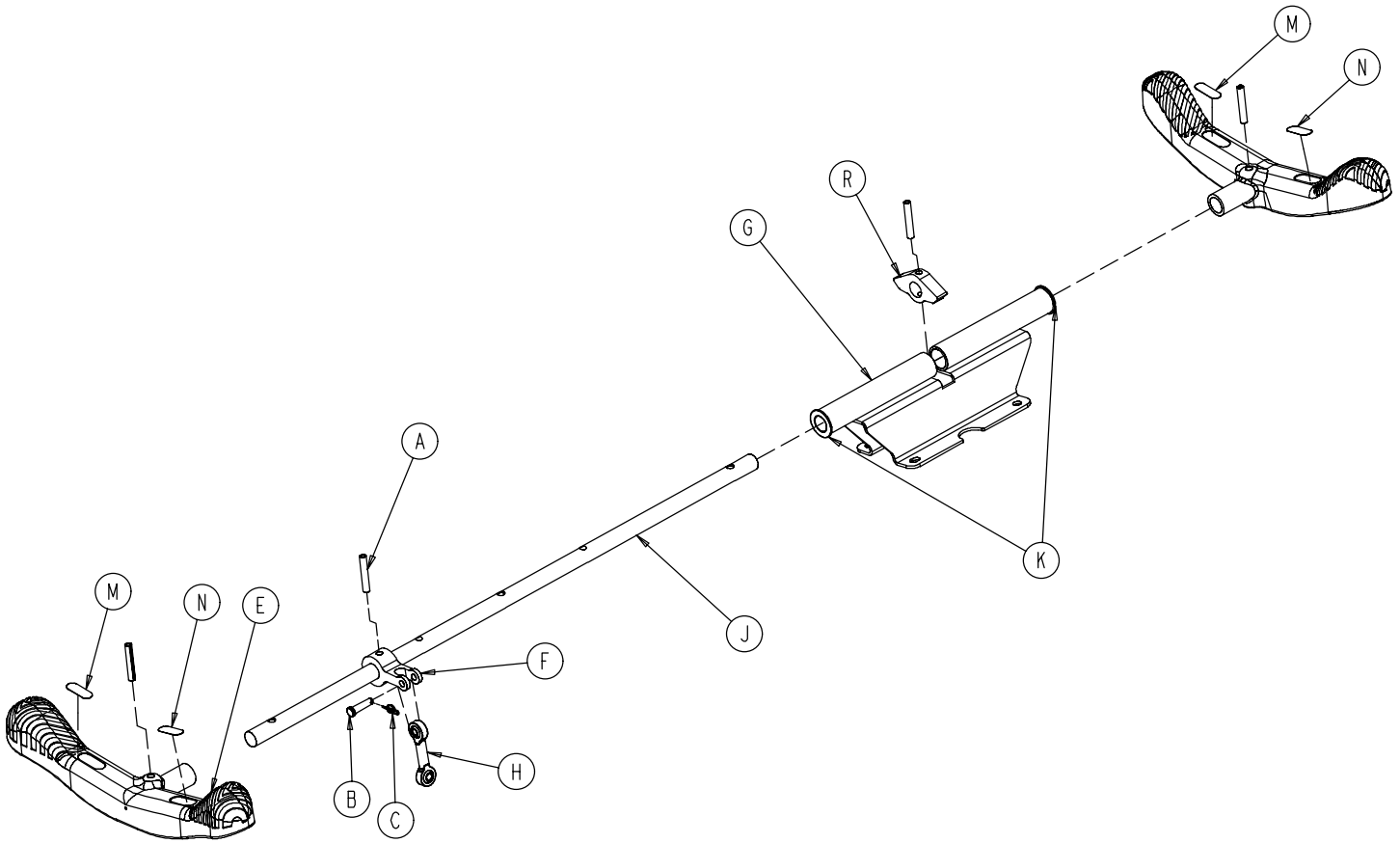


A	0026-067-000	4
B	0026-340-000	1
C	0027-020-000	1
E	0753-003-099	2
F	0753-003-112	1
G	0753-003-114	1
H	0753-003-117	1
J	0753-003-119	1
K	0753-005-044	2
M	1210-201-335	2
N	1210-201-336	2
R	0753-003-132	1


0753-003-125

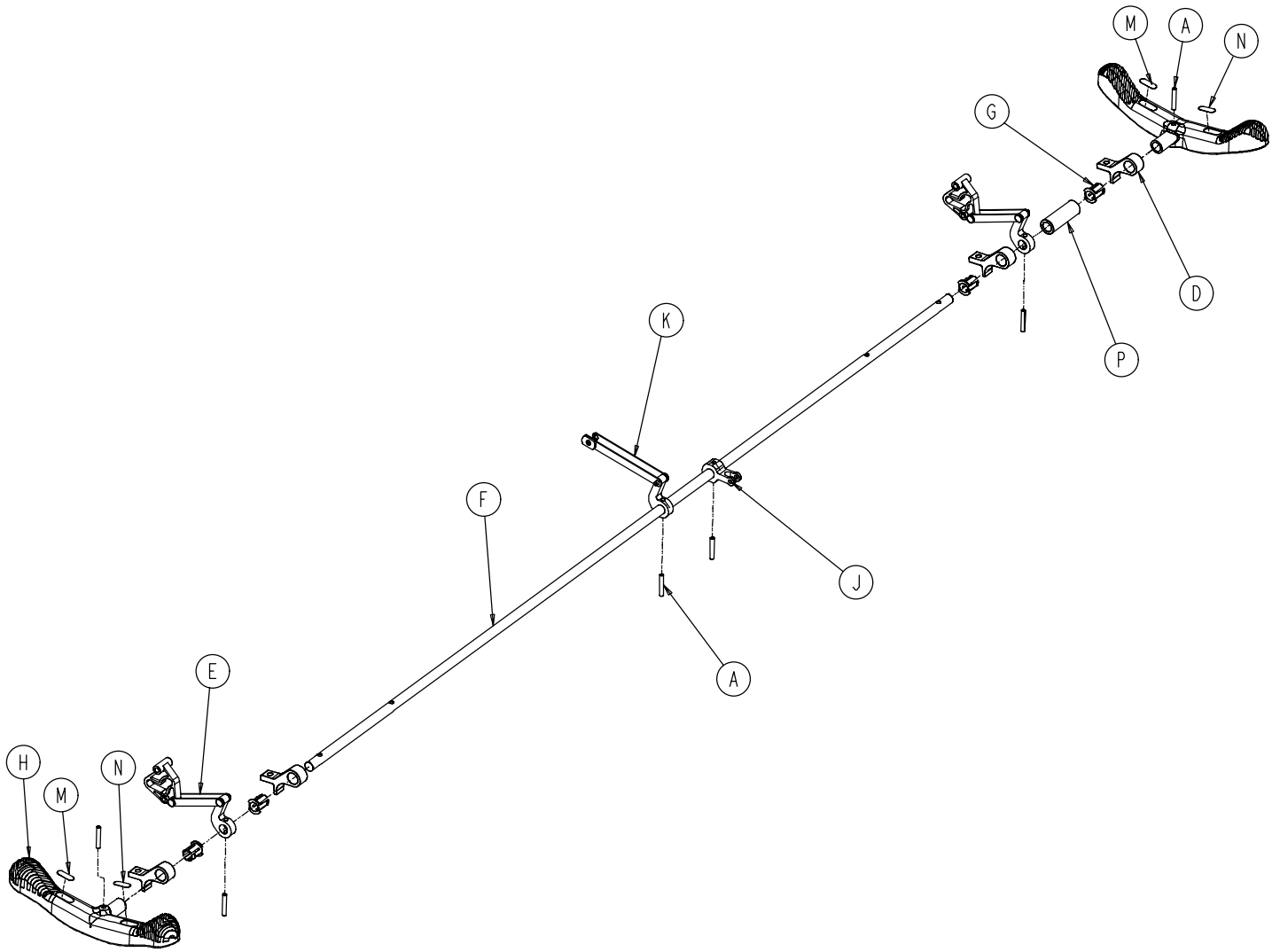



A	0026-067-000	6
D	0753-003-004	4
E	0753-003-010	2
F	0753-003-014	1
G	0753-003-015	4
H	0753-003-099	2
J	0753-003-112	1
K	0753-006-135	1
M	1210-201-335	2
N	1210-201-336	2
P	0753-003-133	1

0753-005-100

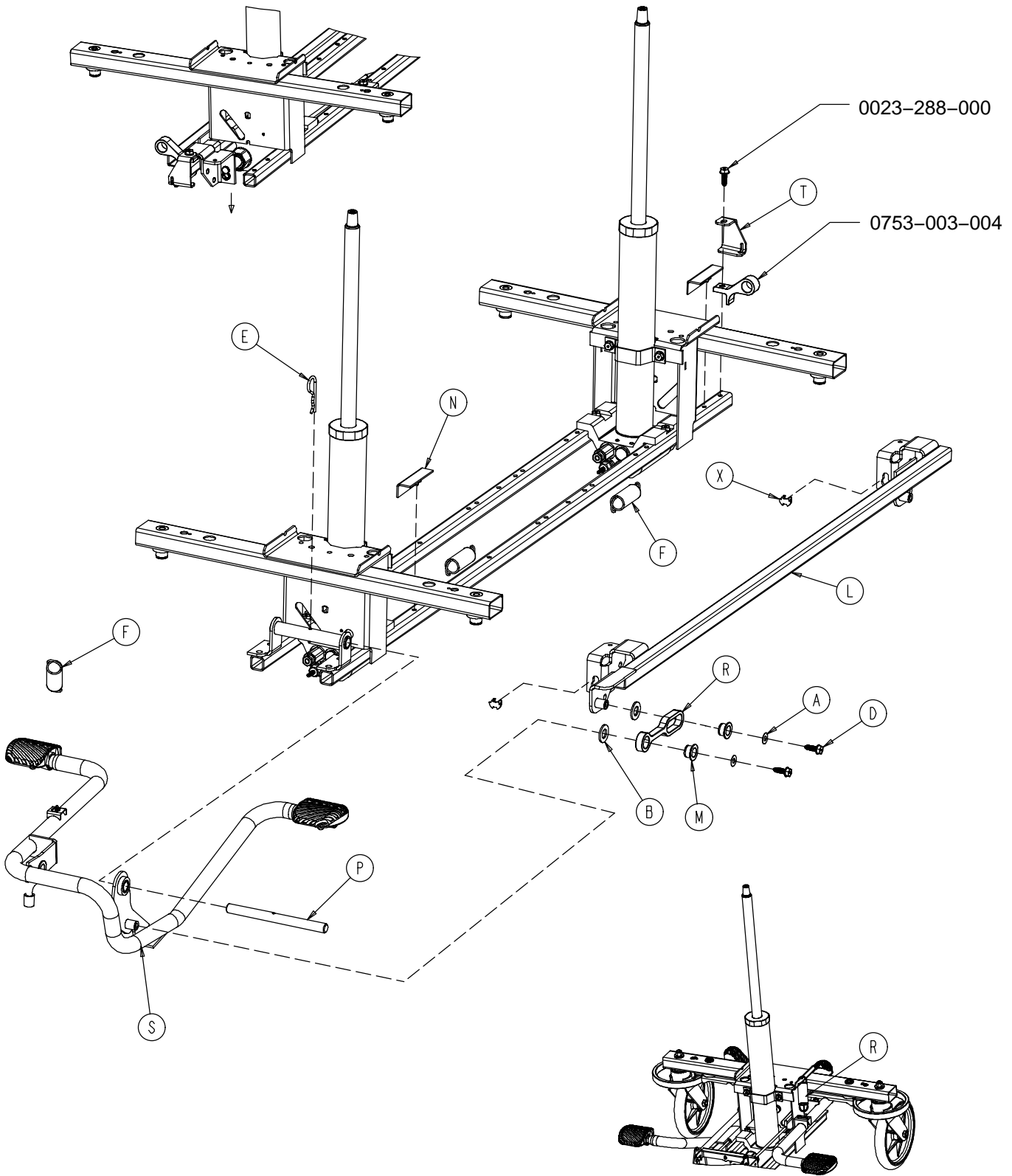


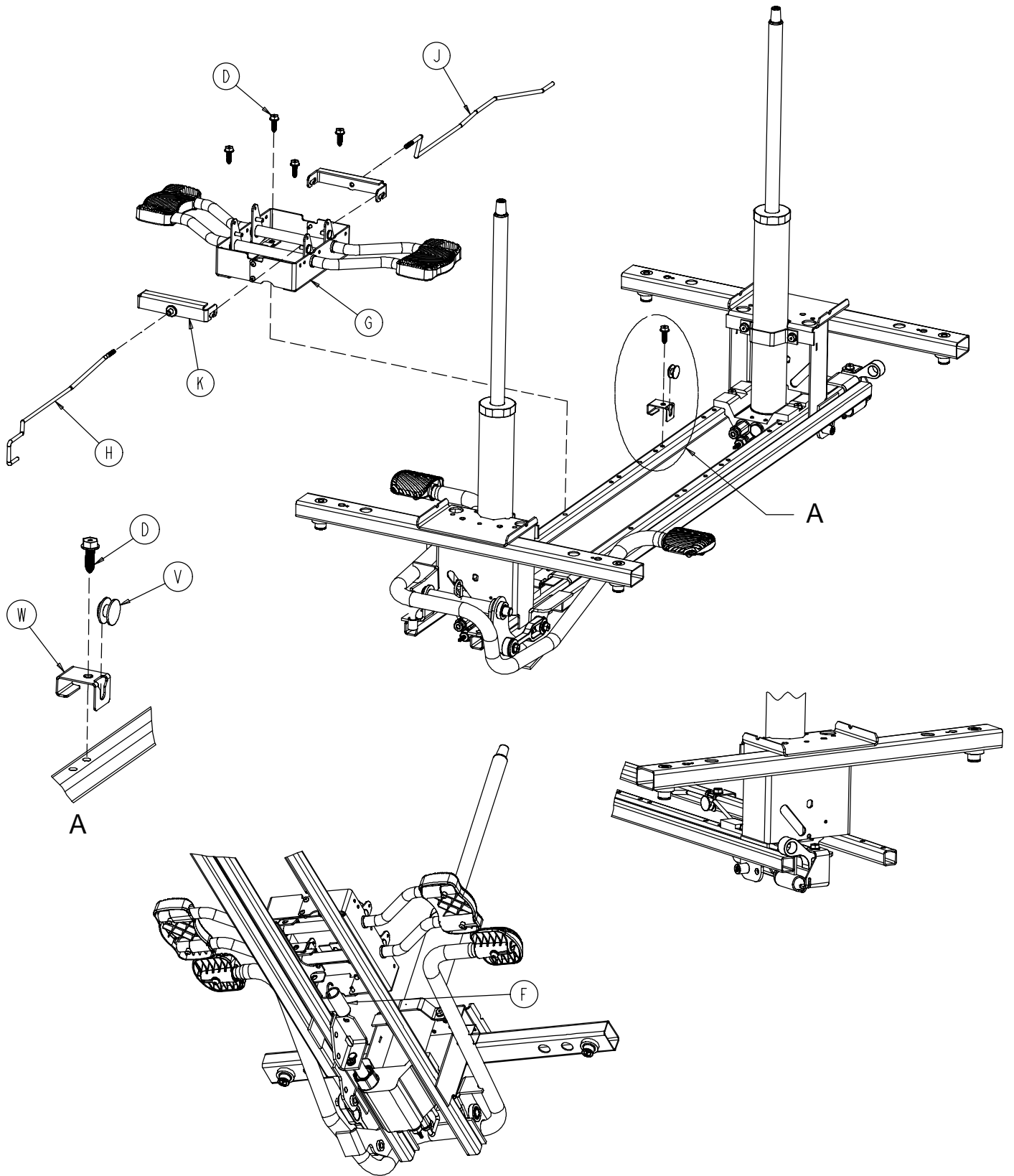

0753-005-100

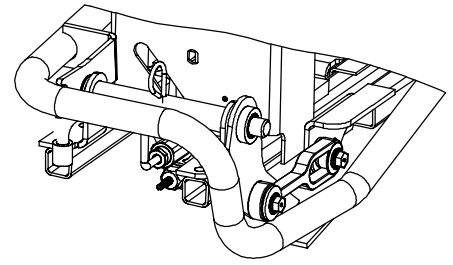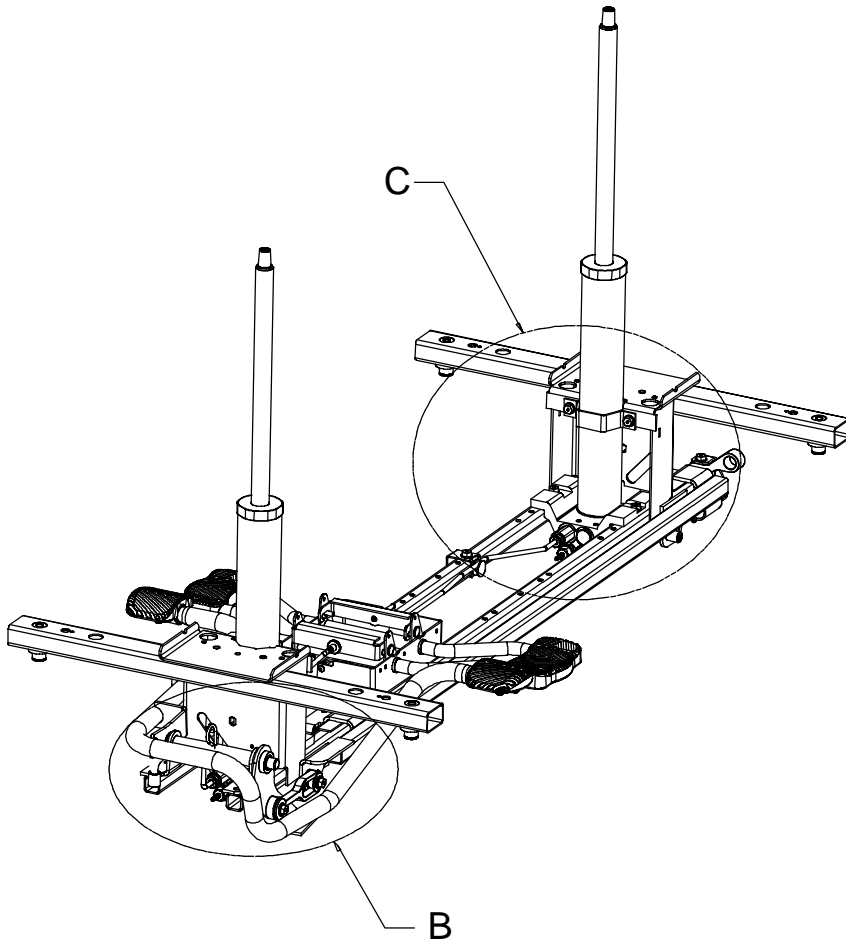


B

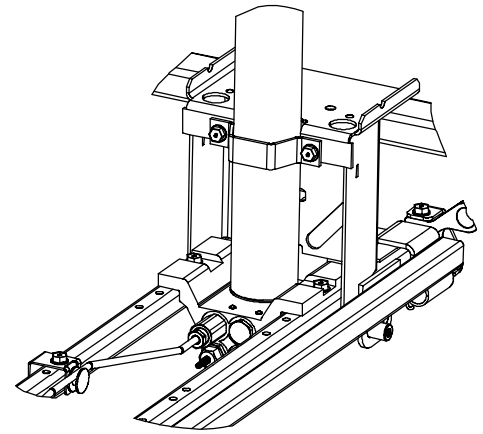

C


A	0011-023-000	2
B	0014-071-000	2
D	0023-288-000	7
E	0027-031-000	1
F	0038-497-000	3
G	0753-004-001	1
H	0753-004-014	1
J	0753-004-015	1
K	0753-004-032	2
L	0753-005-037	1
M	0753-005-044	2
N	0753-005-088	2
P	0753-005-074	1
R	0753-005-075	1
S	0753-005-085	1
T	0753-005-087	1
V	0753-010-015	1
W	0753-010-115	1
X	0753-005-089	2

0753-004-001

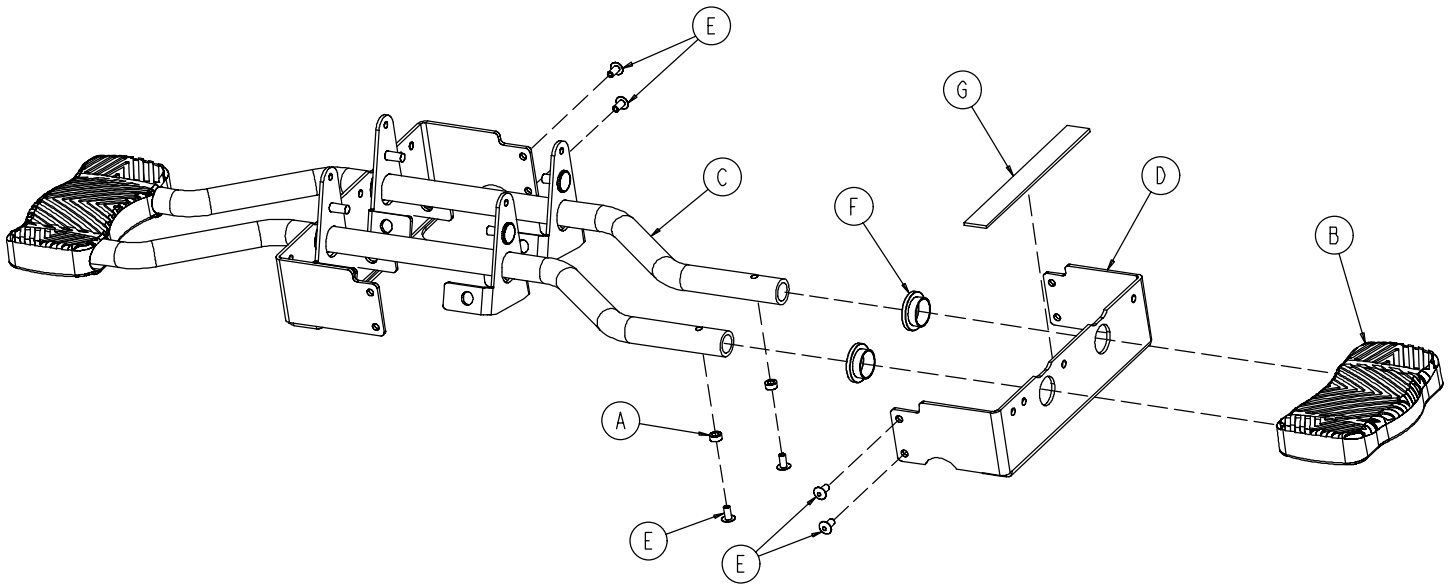



A	0753-004-029	4
B	1061-201-127	2
C	0753-004-004	2
D	0753-004-121	2
E	0025-079-000	8
F	0753-004-036	4
G	0753-004-035	2

0753-005-085

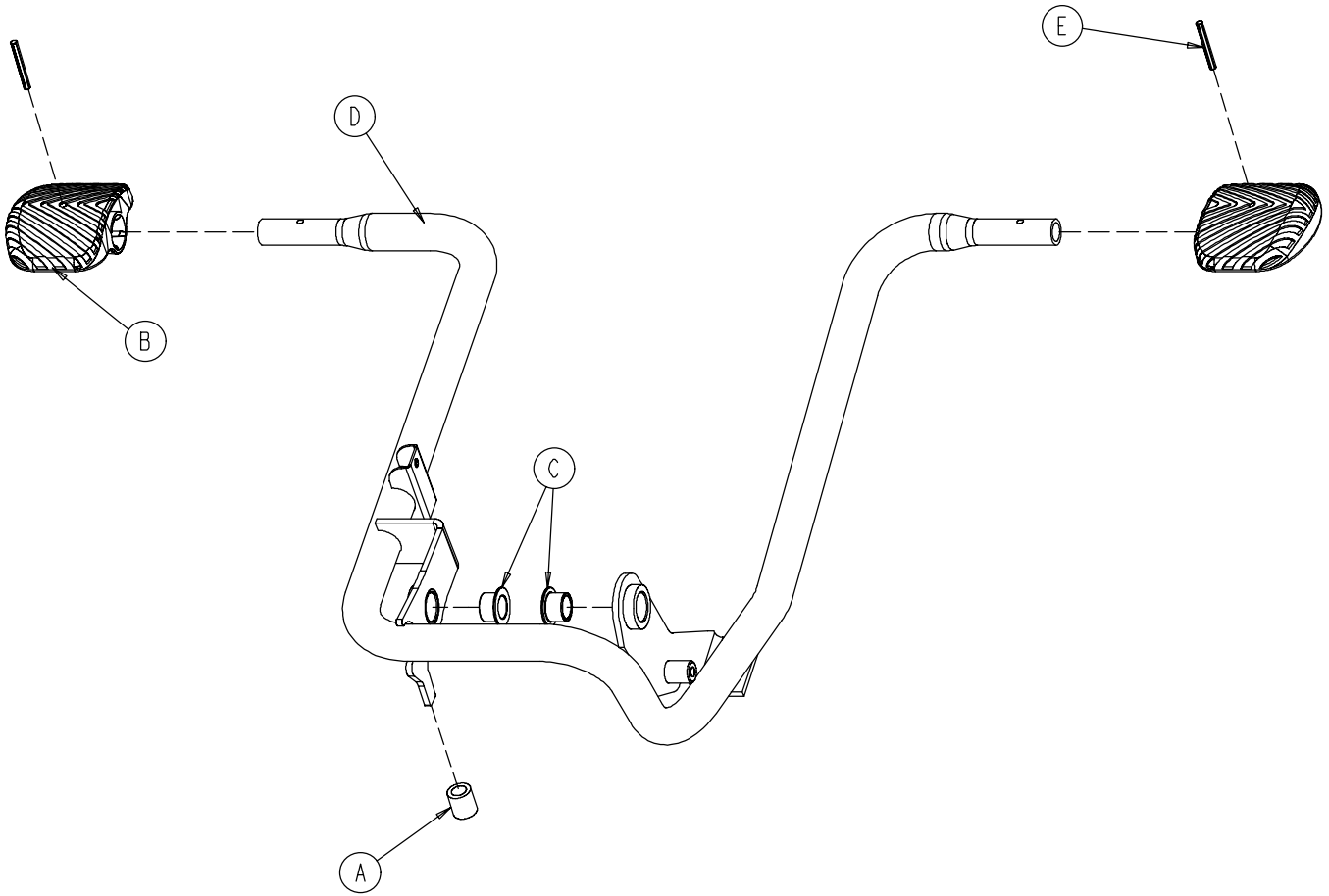

A	0715-001-140	1
B	0715-201-126	2
C	0753-005-044	2
D	0753-005-080	1
E	0026-143-000	2

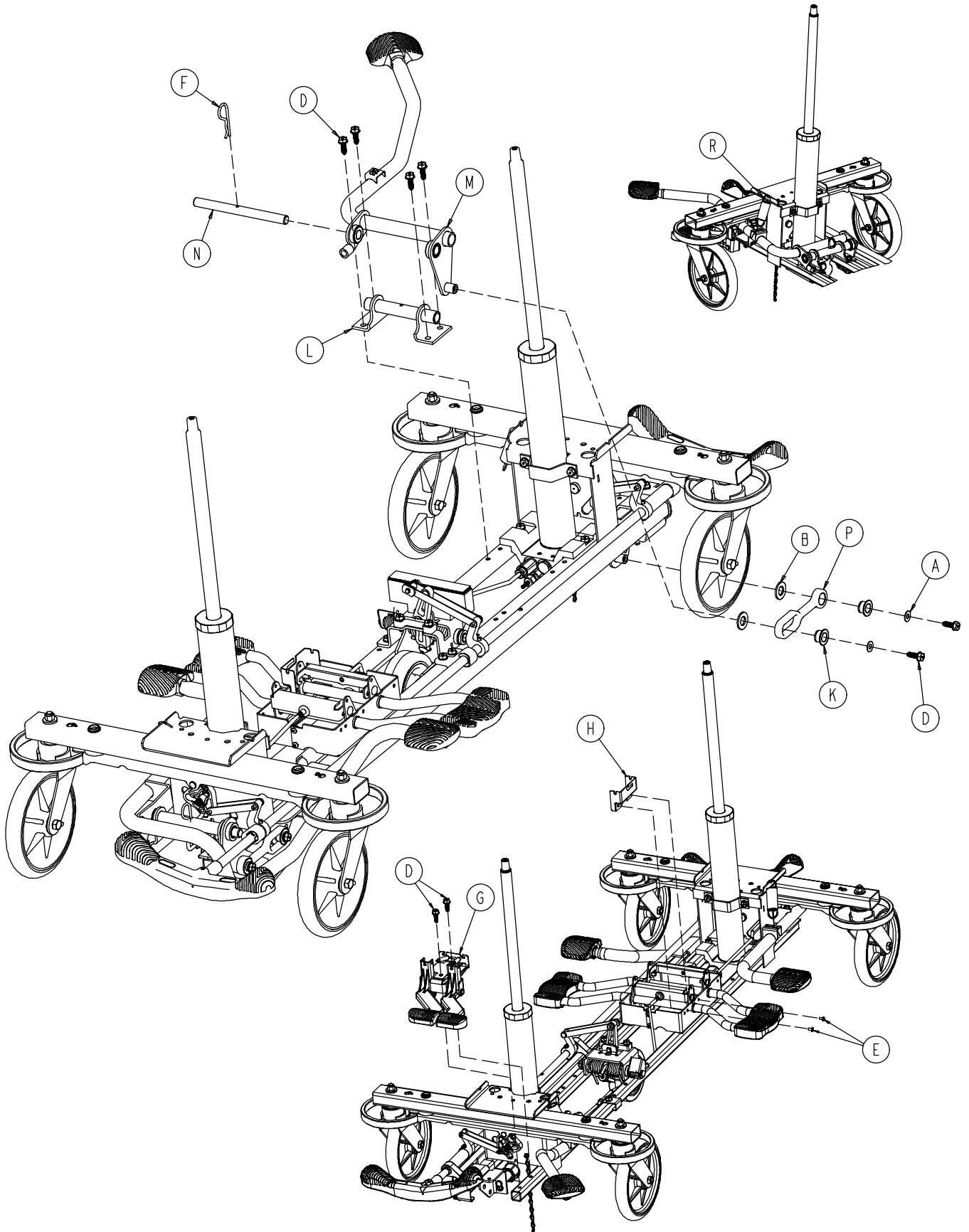


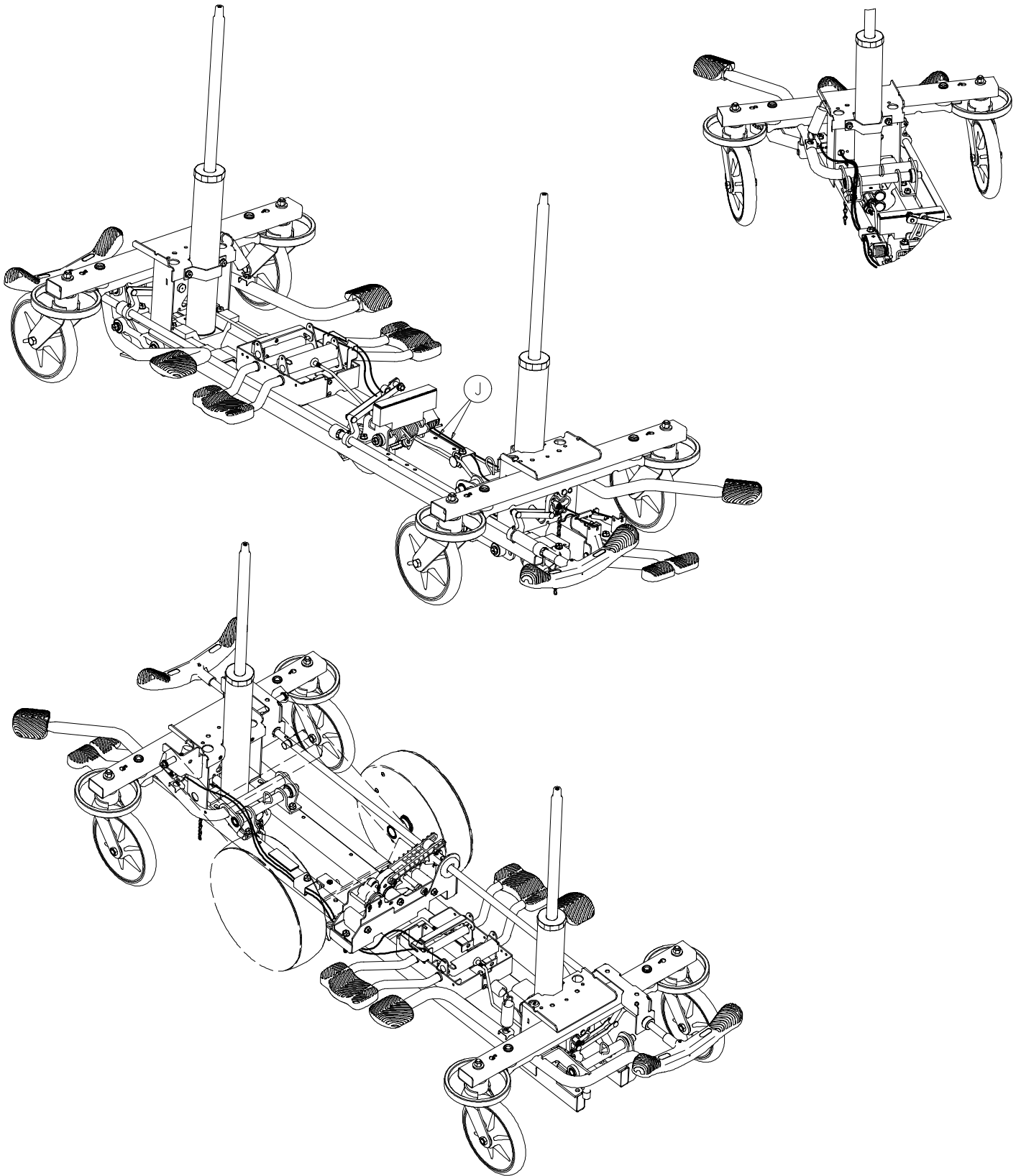

0753-005-110


A	0011-023-000	2
B	0014-071-000	1
D	0023-288-000	8
E	0025-079-000	2
F	0027-031-000	1
G	0753-004-123	1
H	0753-004-127	1
J	0753-004-030	2
K	0753-005-044	2
L	0753-005-063	1
M	0753-005-073	1
N	0753-005-074	1
P	0753-005-075	1
R	0038-497-000	1

0753-004-123

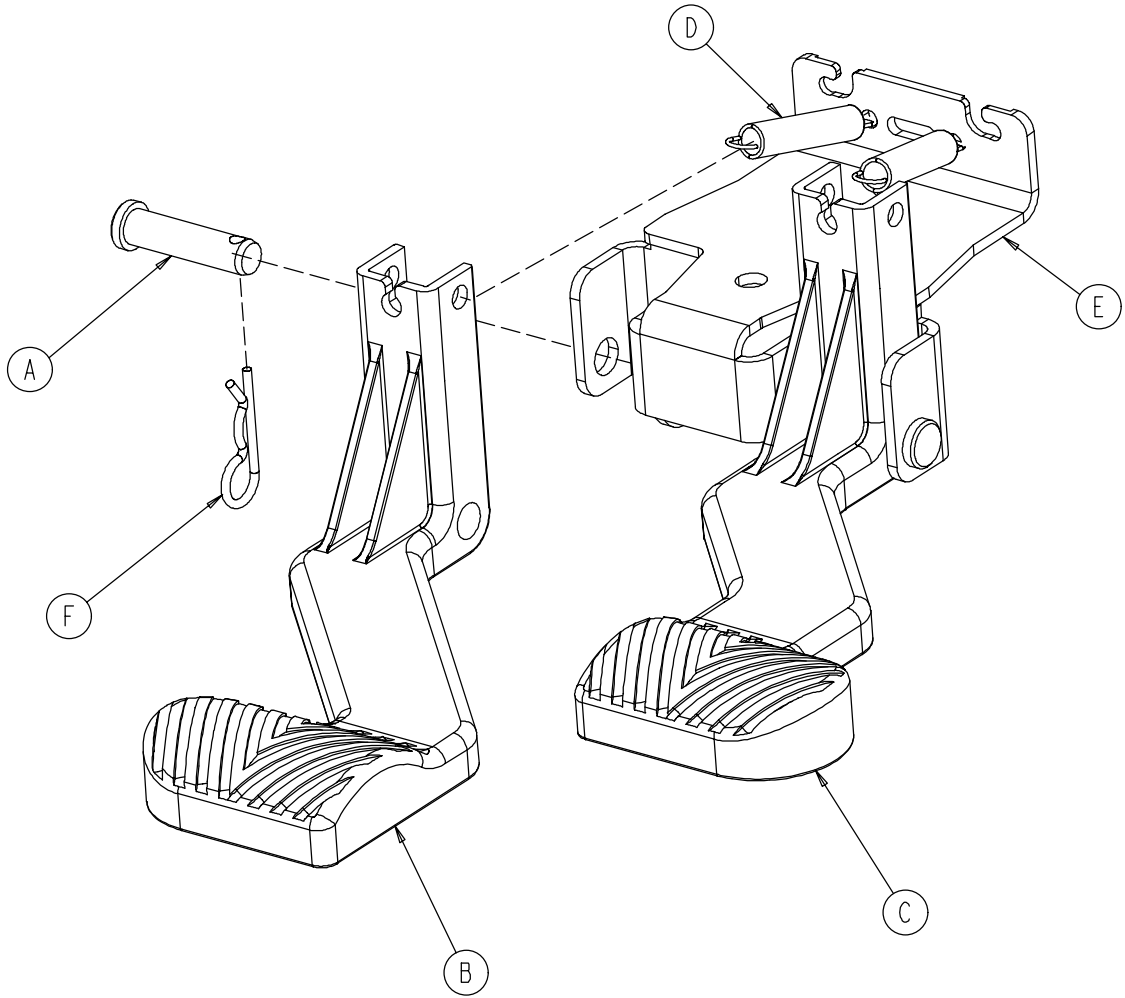


A	0026-132-000	2
B	0753-004-024	1
C	0753-004-025	1
D	0753-004-028	2
E	0753-004-322	1
F	0027-009-000	2

0753-005-073

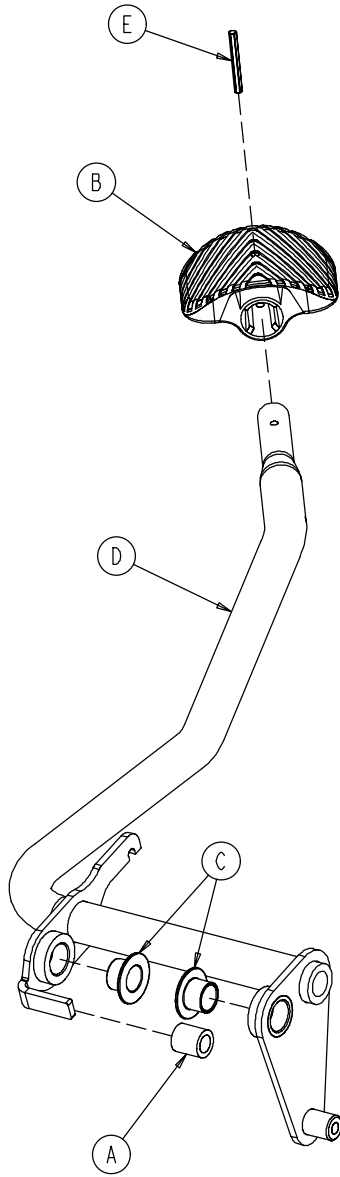


A	0715-001-140	1
B	0715-201-126	1
C	0753-005-044	2
D	0753-005-071	1
E	0026-143-000	1

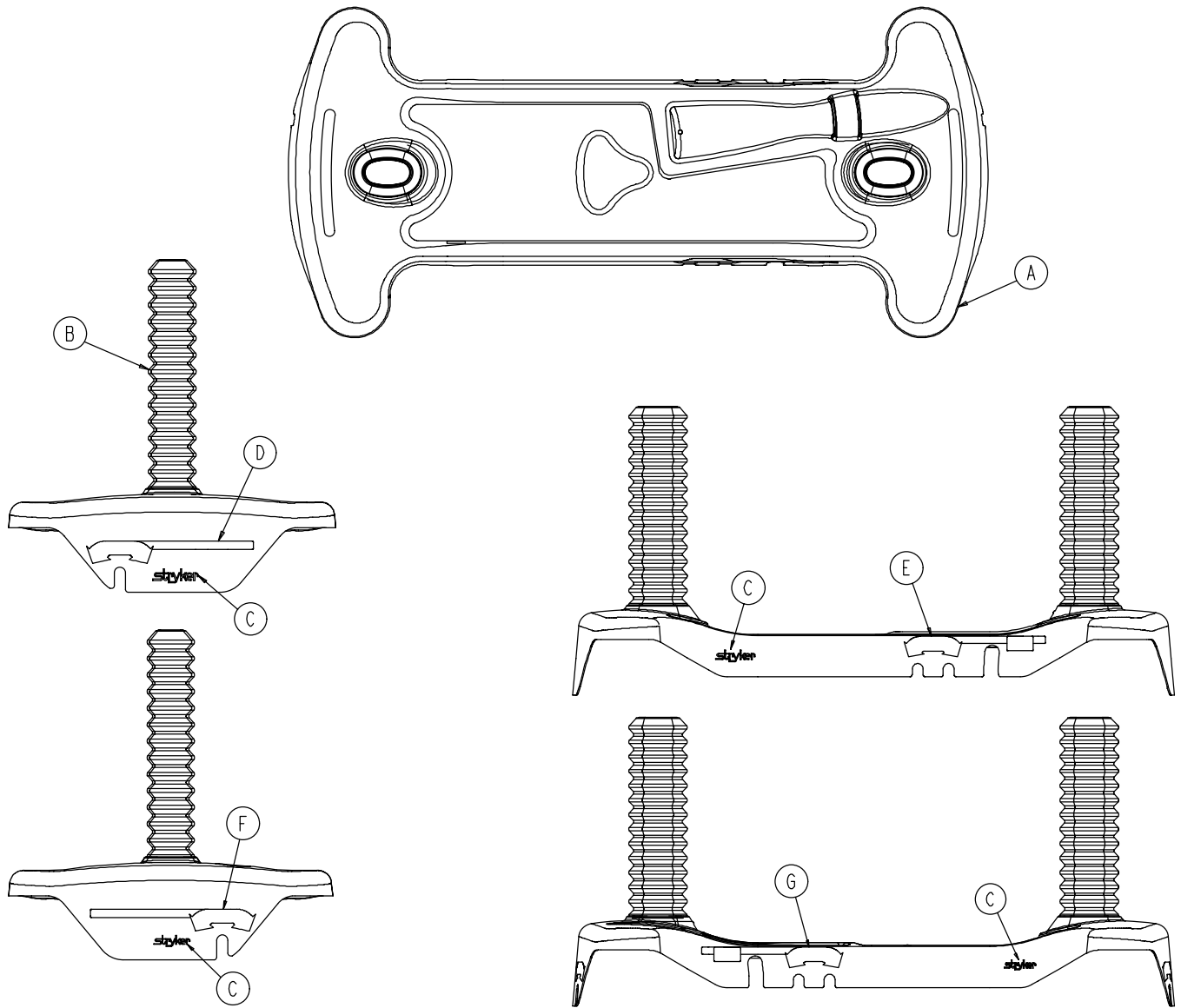

A	0753-010-040	1
B	1040-010-134	2
C	0946-201-060	4


0753-010-063

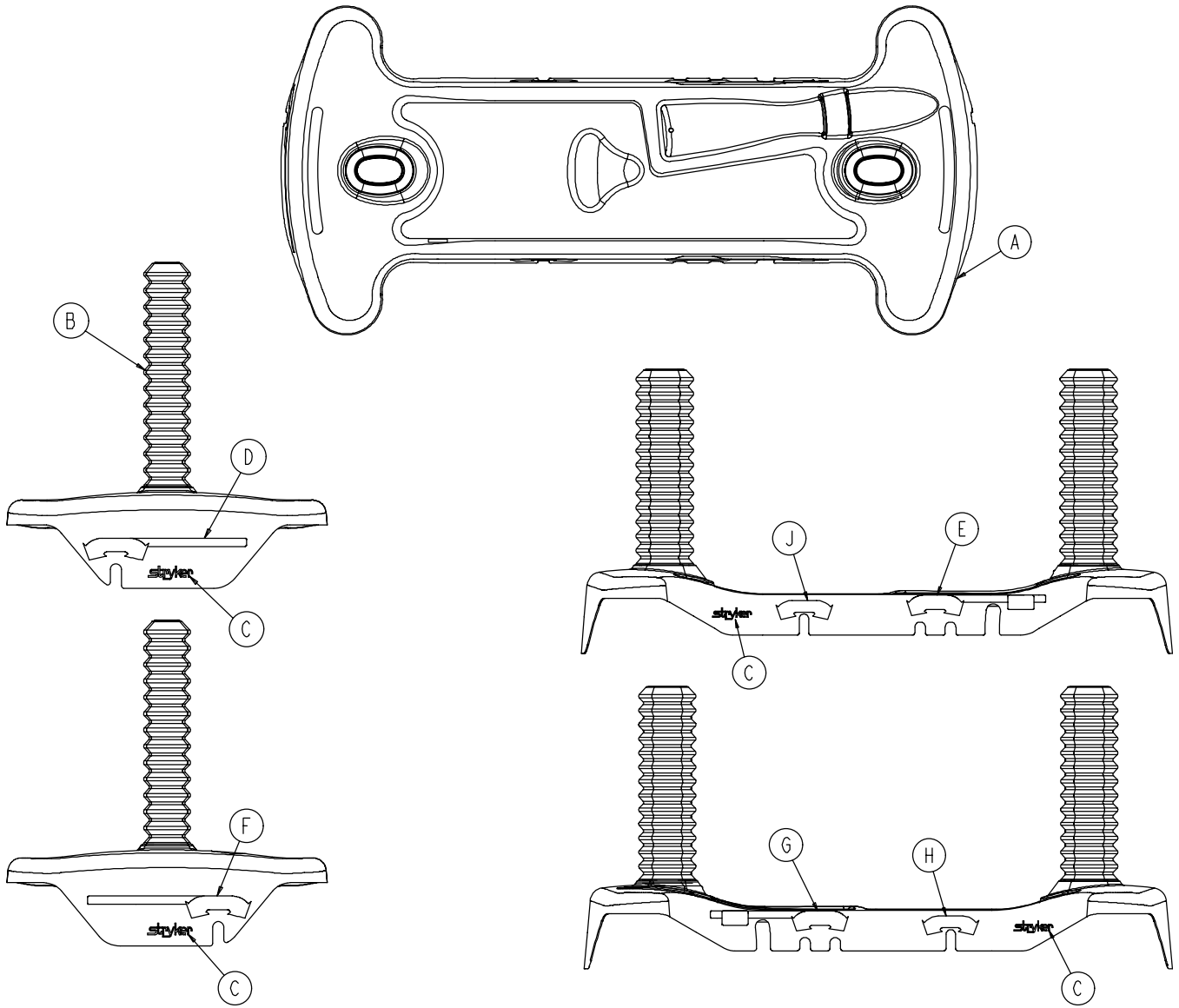


A	0753-010-040	1
B	1040-010-134	2
C	0946-201-060	4
H	0753-010-018	1
J	0753-010-019	1


Labels

	D	E	F	G
RED 0753-020-120	0753-010-091	0753-010-055	0753-010-057	0753-010-054
PURPLE 0753-020-121	0753-010-191	0753-010-155	0753-010-210	0753-010-154
GREEN 0753-020-122	0753-010-291	0753-010-255	0753-010-310	0753-010-254
GRAY 0753-020-123	0753-010-391	0753-010-355	0753-010-410	0753-010-354
TEAL 0753-020-124	0753-010-491	0753-010-455	0753-010-510	0753-010-454
PINK 0753-020-125	0753-010-591	0753-010-555	0753-010-610	0753-010-554
BLUE 0753-020-126	0753-010-691	0753-010-655	0753-010-710	0753-010-654

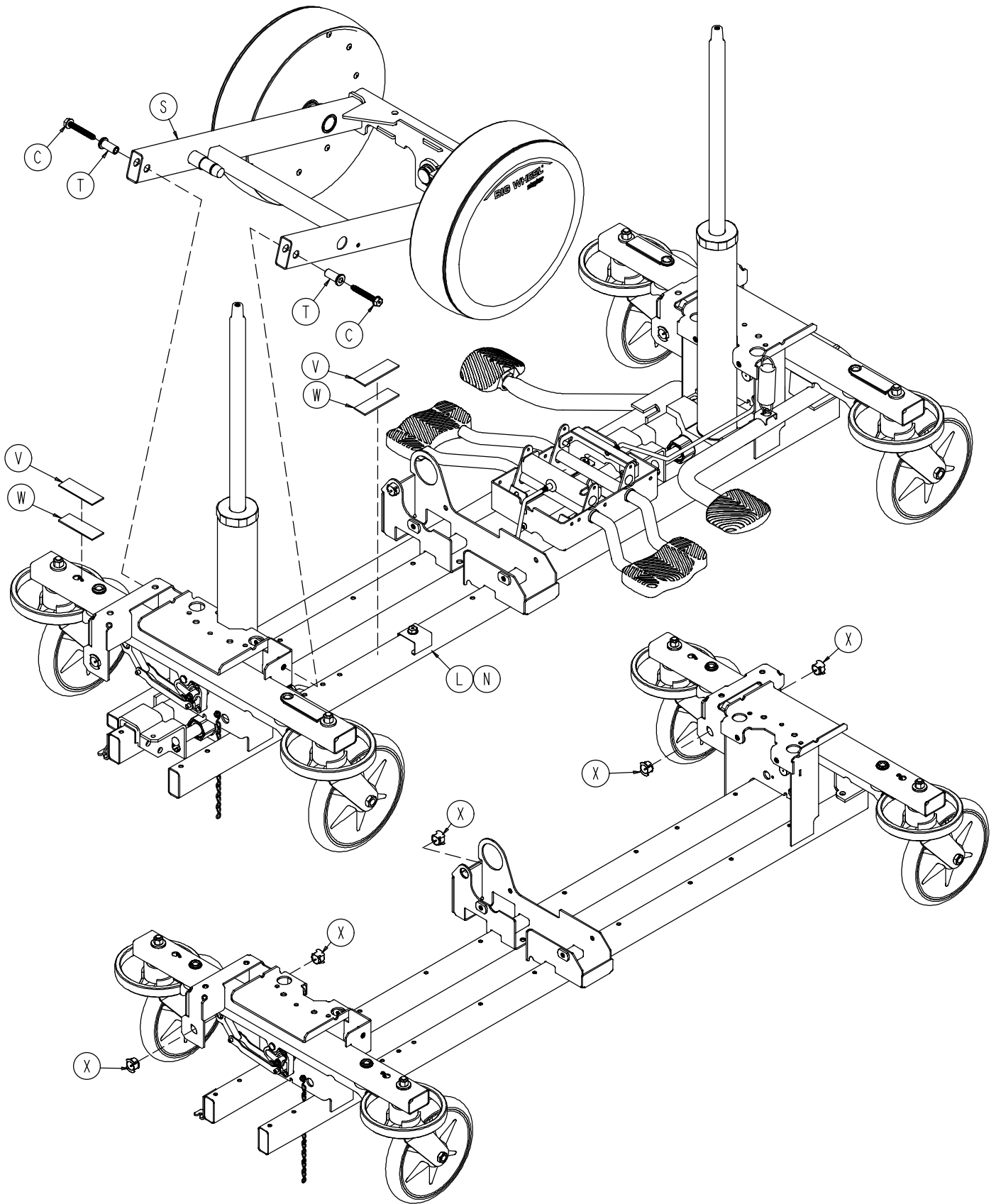


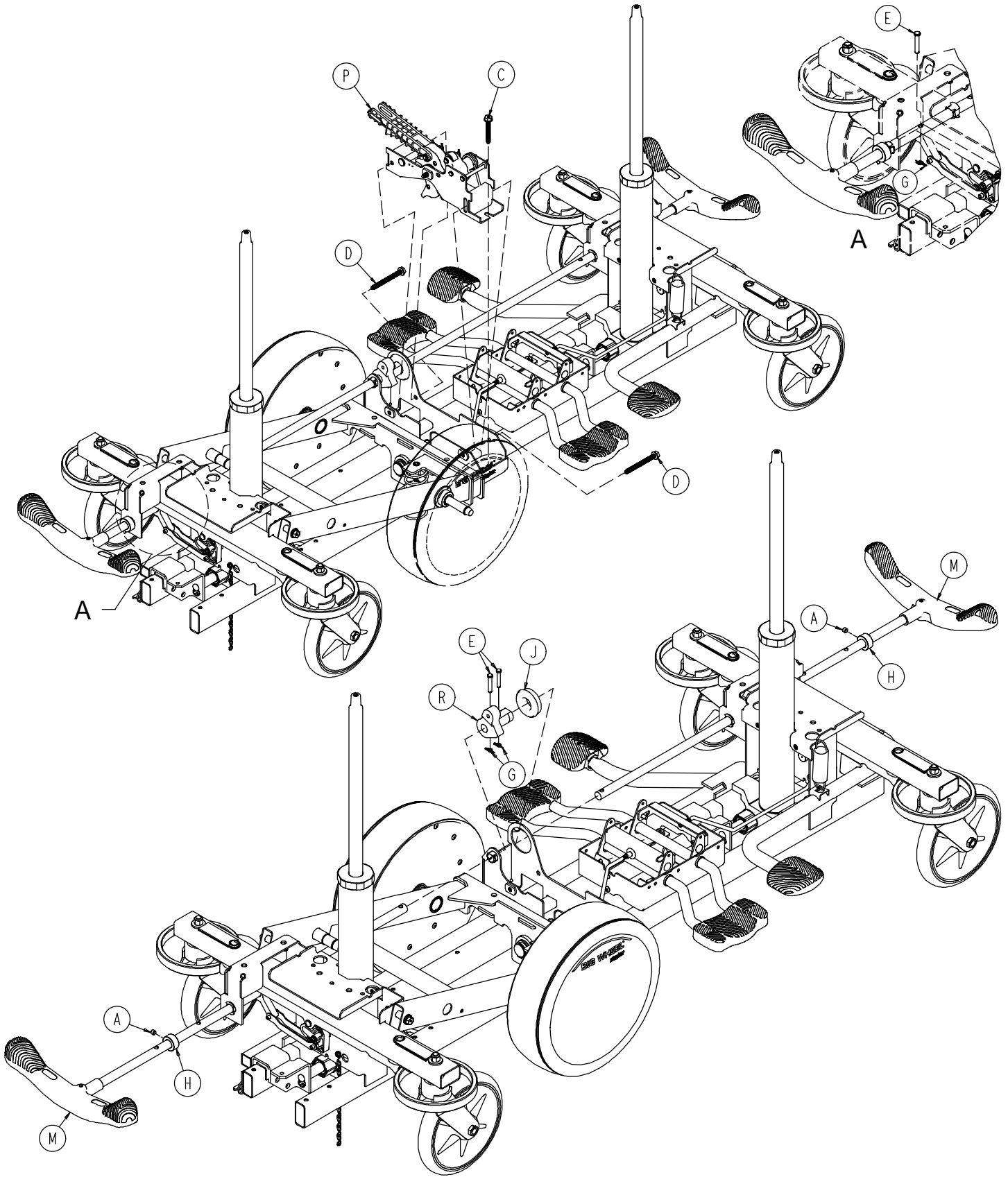


1040-015-005

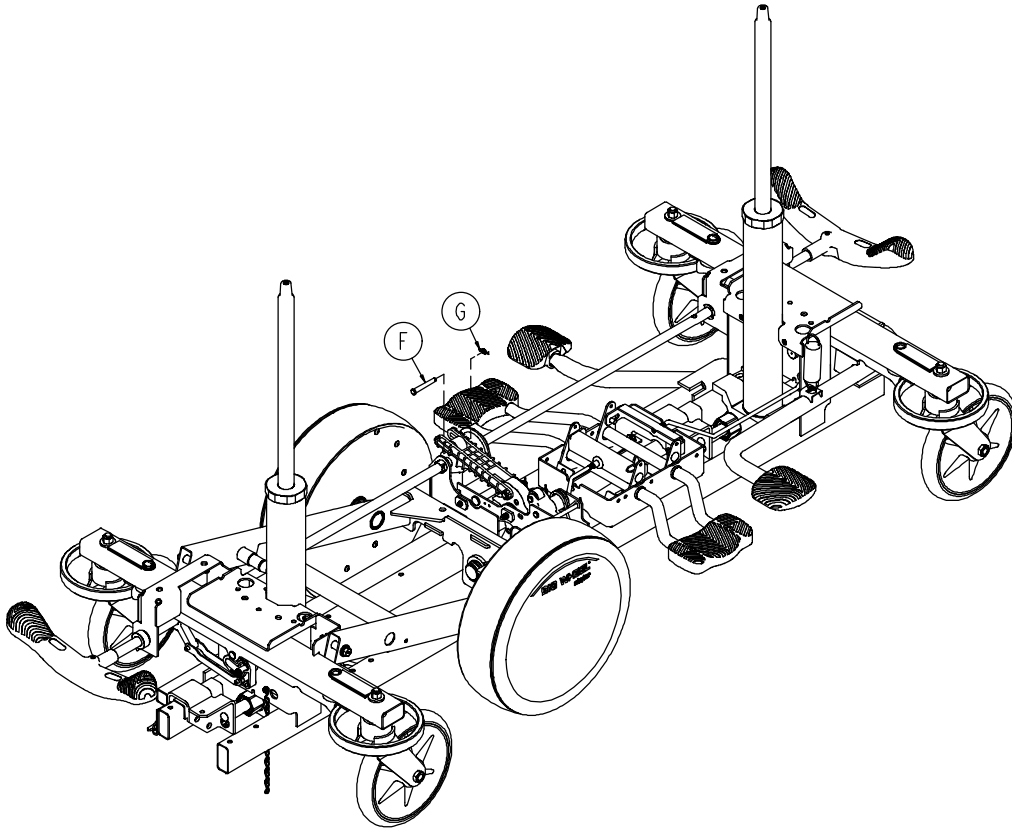

A	0021-166-000	2
C	0023-150-000	3
D	0023-151-000	2
E	0026-273-000	4
F	0025-330-000	1
G	0027-020-000	5
H	0042-020-000	2
J	0081-330-000	1
L	1040-003-105	1
M	1040-003-191	2
N	1040-005-200	1
P	1040-006-001	1
R	1040-006-131	1
S	1040-015-001	1
T	1040-015-007	2
V	0029-007-000	5
W	0029-009-000	5
X	1210-201-251	5

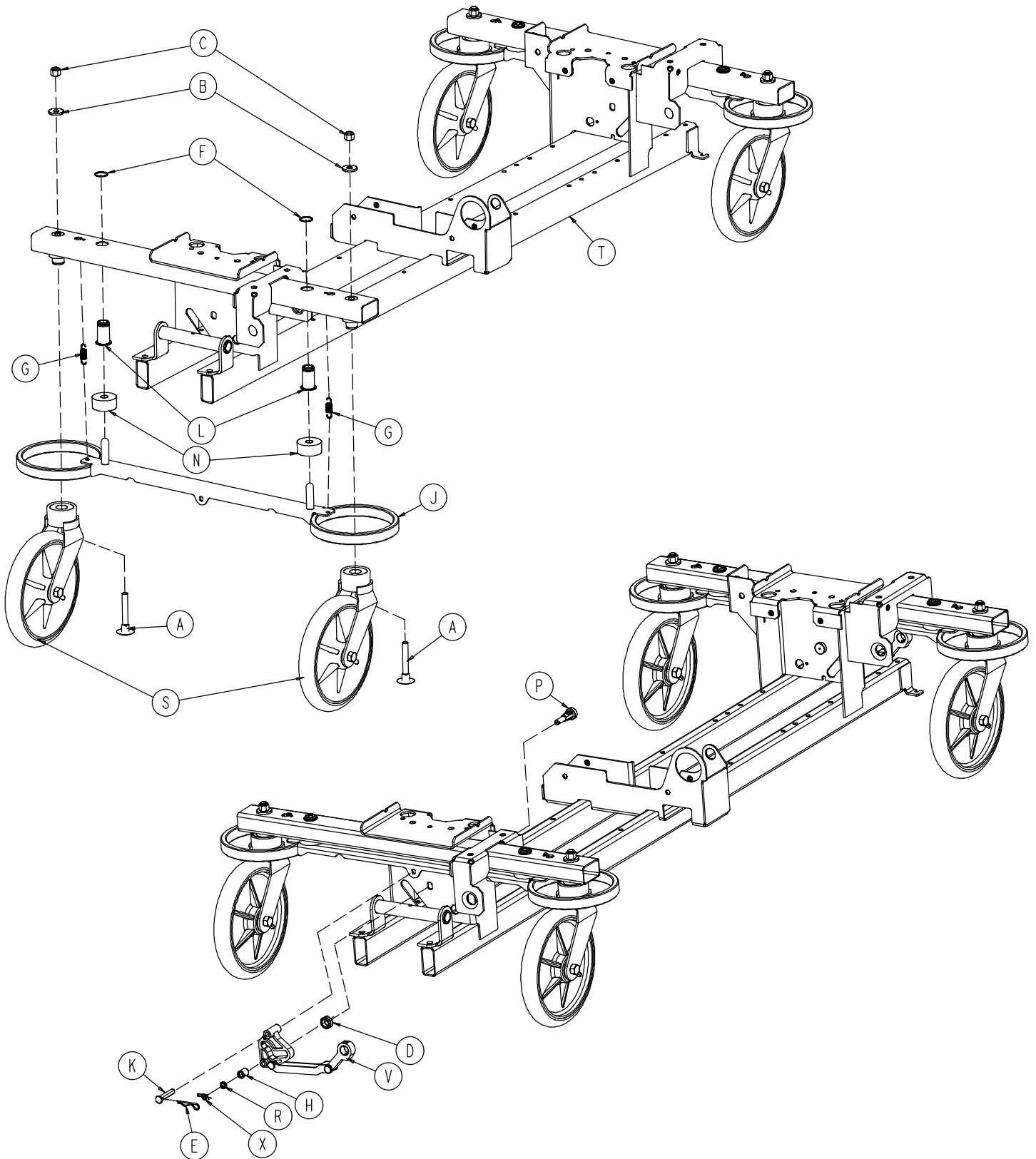


1040-003-105

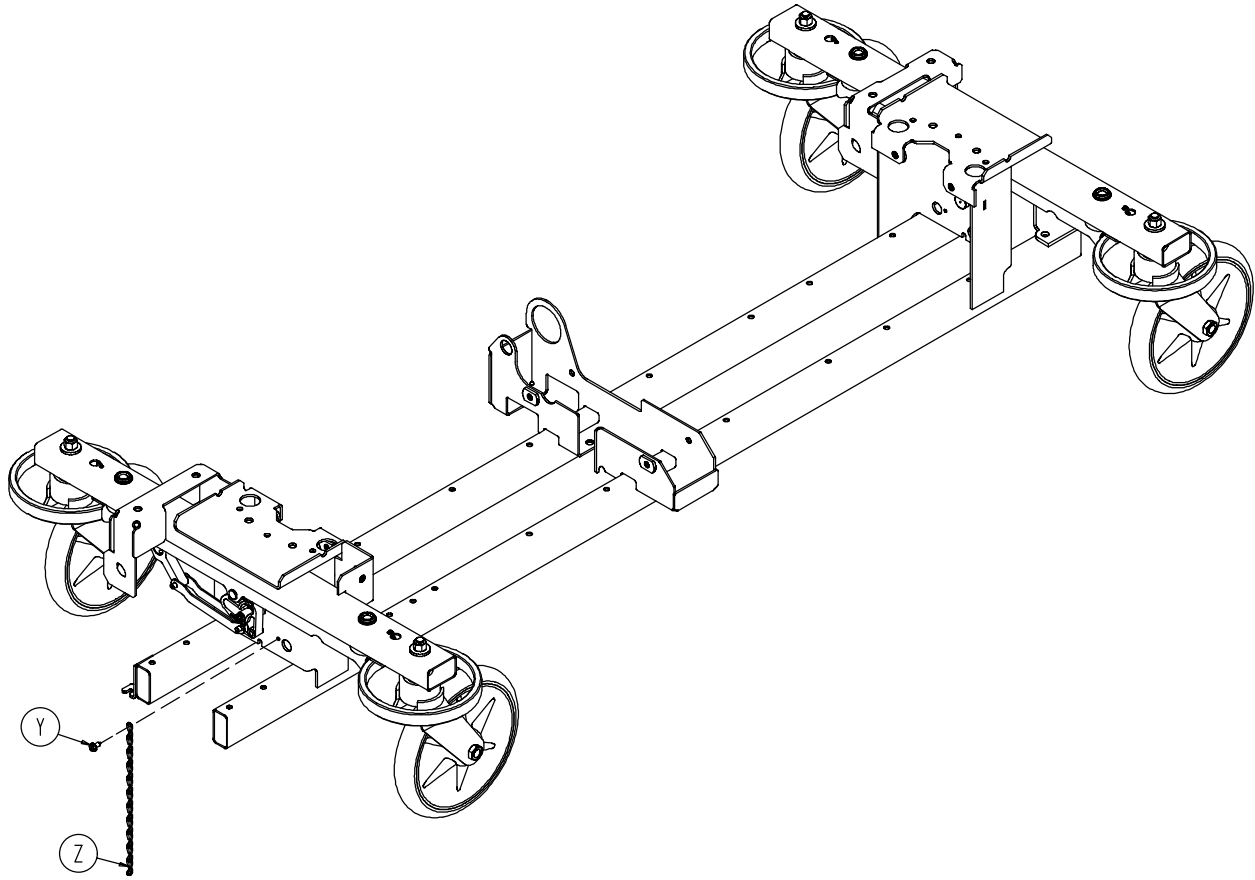


A	0005-039-000	4
B	0011-262-000	4
C	0016-035-000	4
D	0016-049-000	2
E	0027-012-000	2
F	0028-037-000	4
G	0038-439-000	4
H	0081-272-000	2
J	0753-003-006	2
K	0753-003-066	2
L	0753-003-079	4
N	0753-003-121	4
P	0753-003-130	2
R	0753-003-131	2
S	0753-010-020	4
T	1040-001-001	1
V	1040-003-085	2
X	0027-019-000	2
Y	0023-025-000	1
Z	0753-010-012	1

1040-003-185

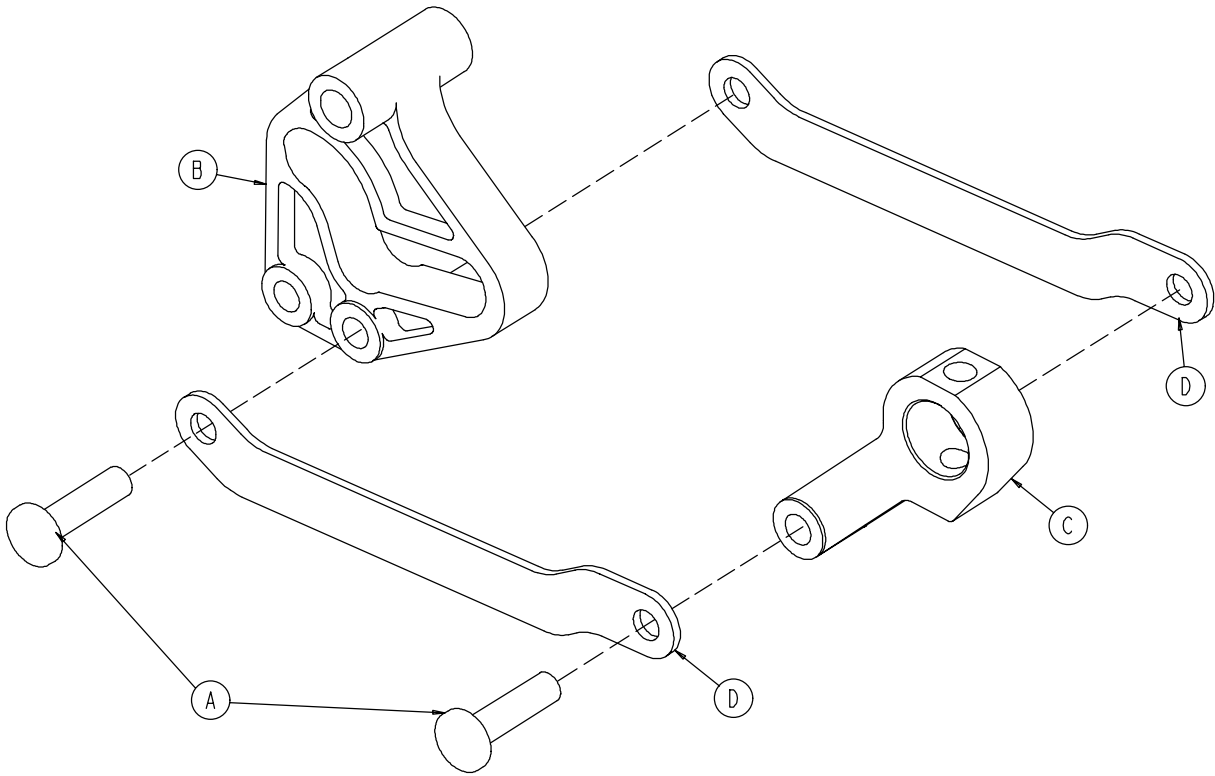



A	0753-003-098	2
B	0753-003-102	1
C	1040-003-174	1
D	1040-003-076	2



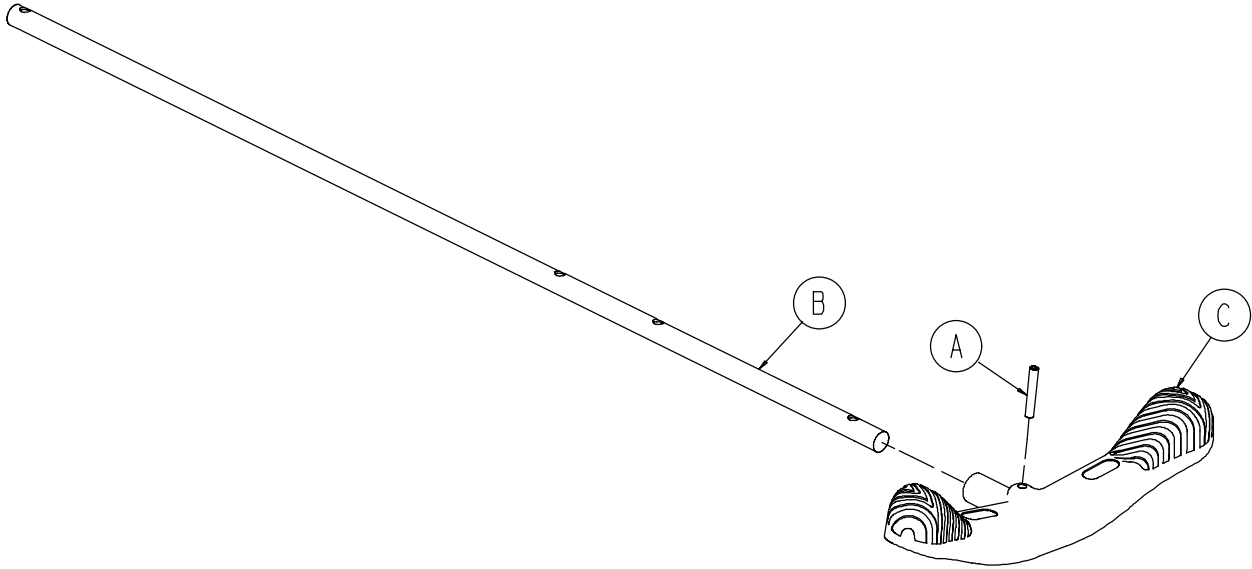


A	0026-067-000	1
B	1040-003-113	1
C	0753-003-099	1

1040-005-200

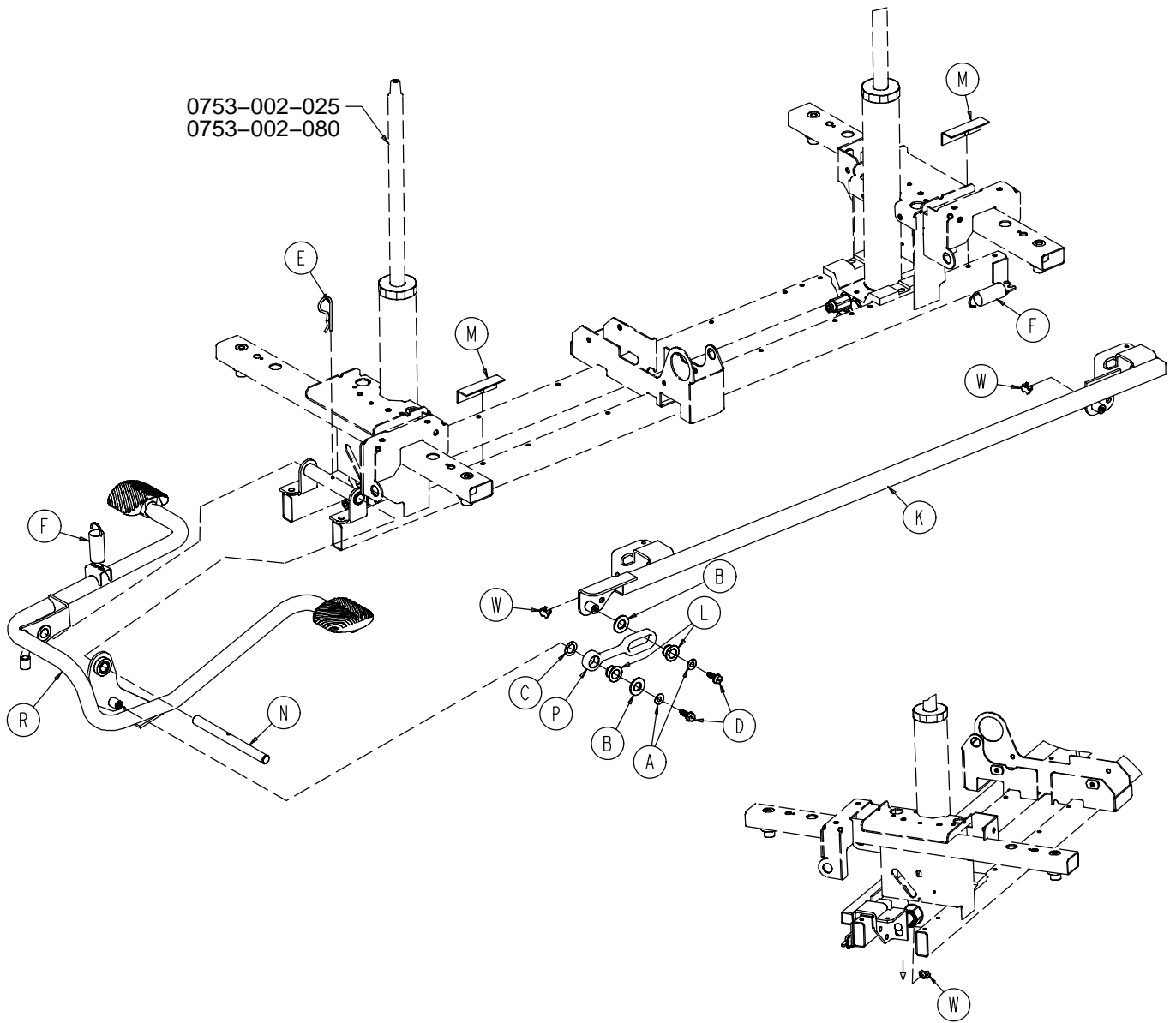


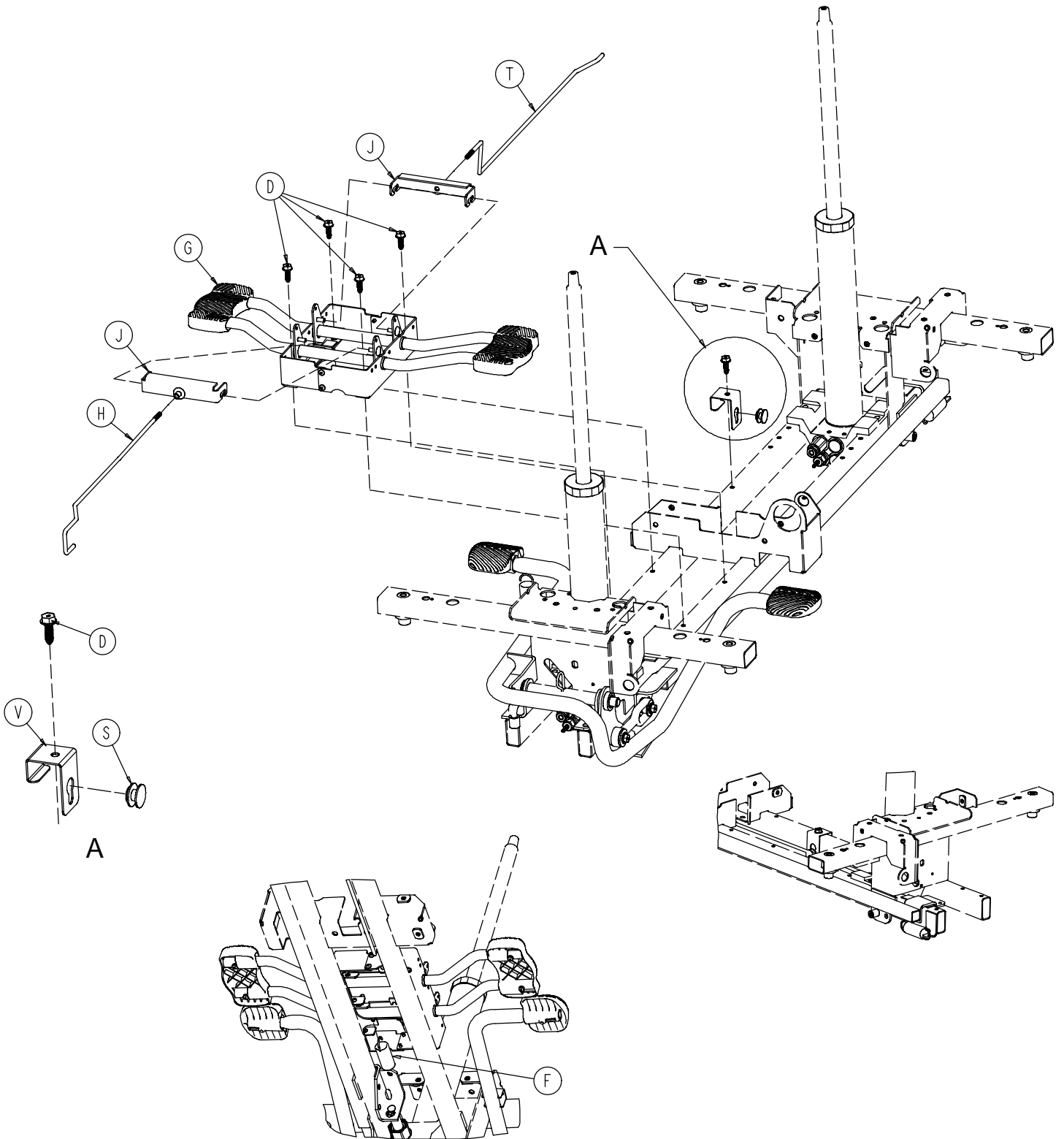

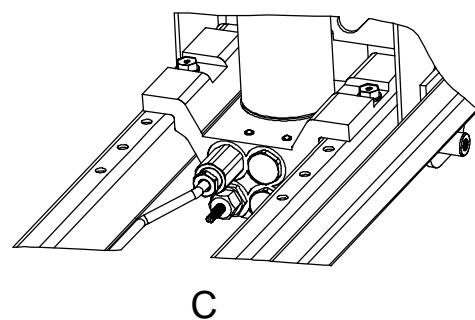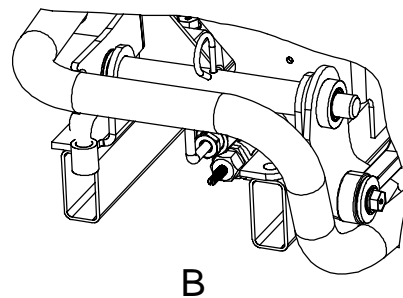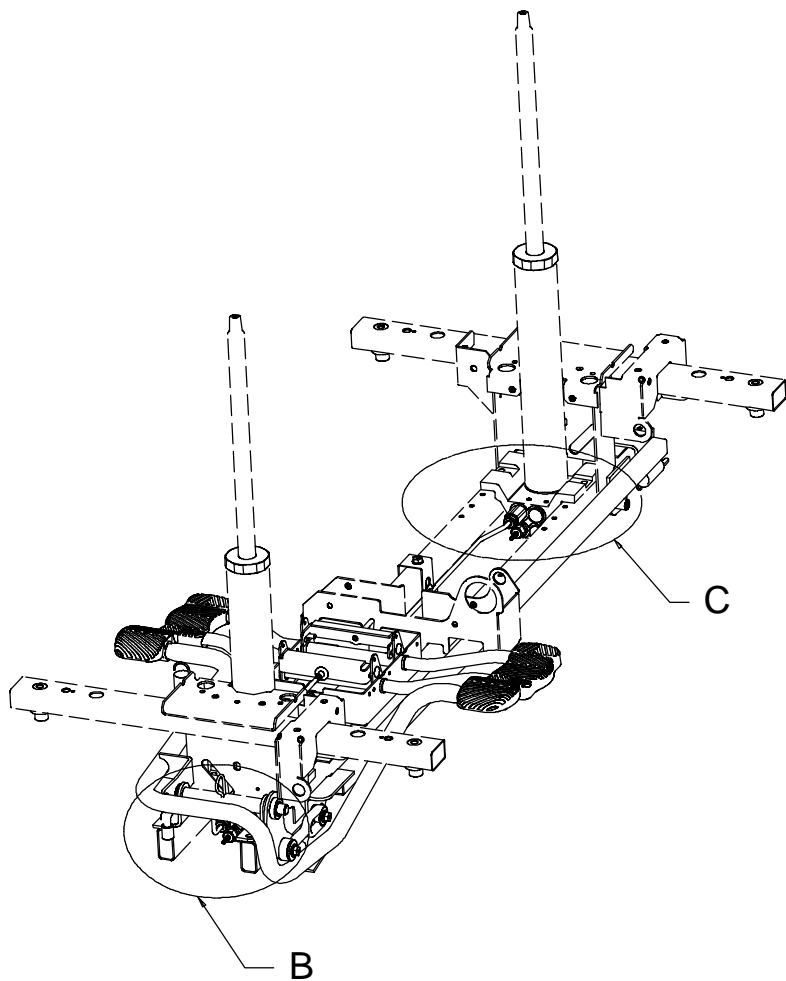


A	0011-023-000	2
B	0014-071-000	2
C	0014-080-000	1
D	0023-288-000	7
E	0027-031-000	1
F	0038-497-000	2
G	0753-004-001	1
H	0753-004-014	1
J	0753-004-032	2
K	0753-005-037	1
L	0753-005-044	2
M	0753-005-088	3
N	0753-005-074	1
P	0753-005-075	1
R	0753-005-085	1
S	0753-010-015	1
T	1040-004-015	1
V	1040-010-115	1
W	0753-005-089	2

0753-004-001

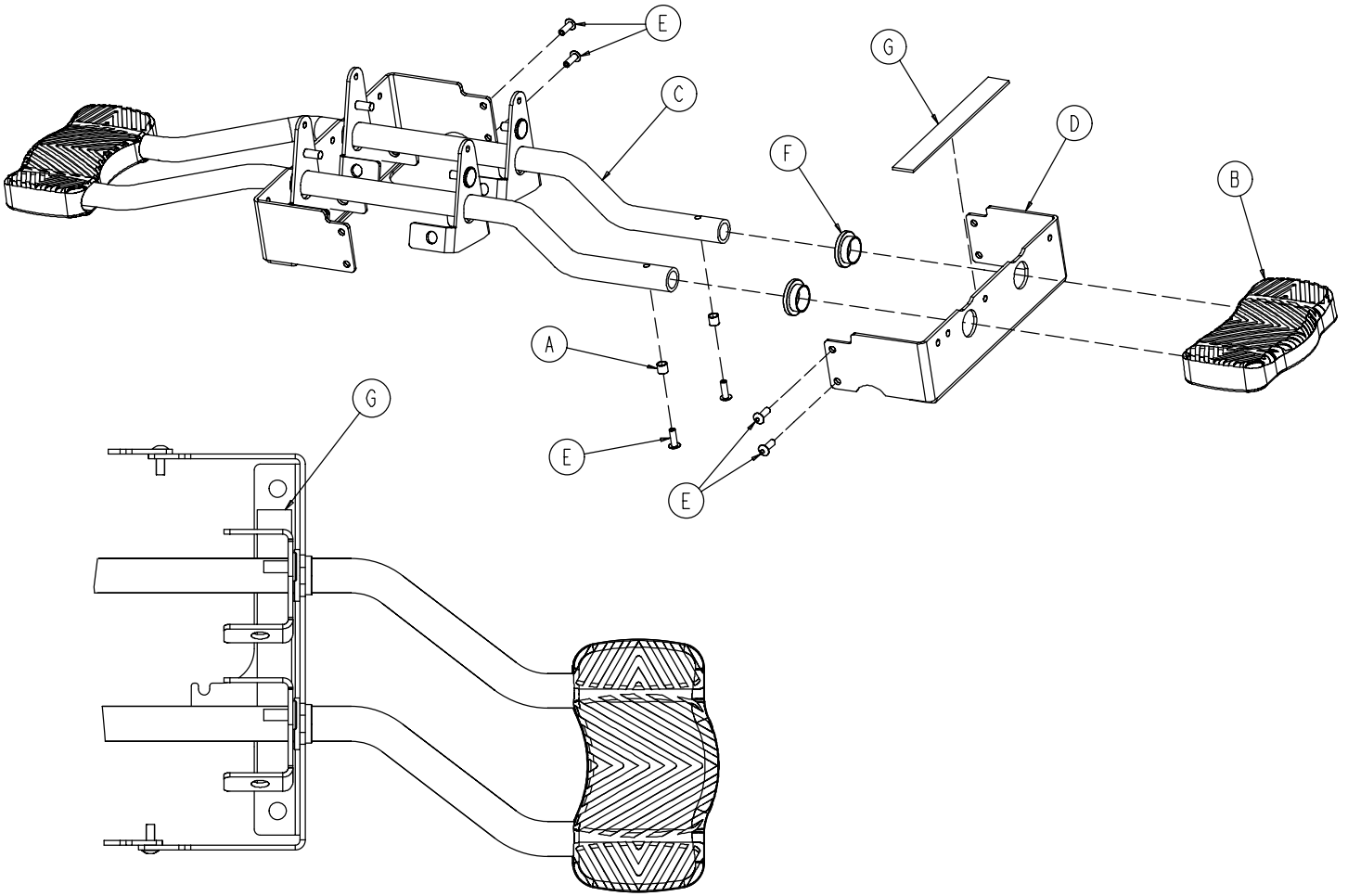


A	0715-001-333	4
B	1061-201-127	2
C	0753-004-004	2
D	0753-004-121	2
E	0025-171-000	8
F	0753-004-036	4
G	0753-004-035	2

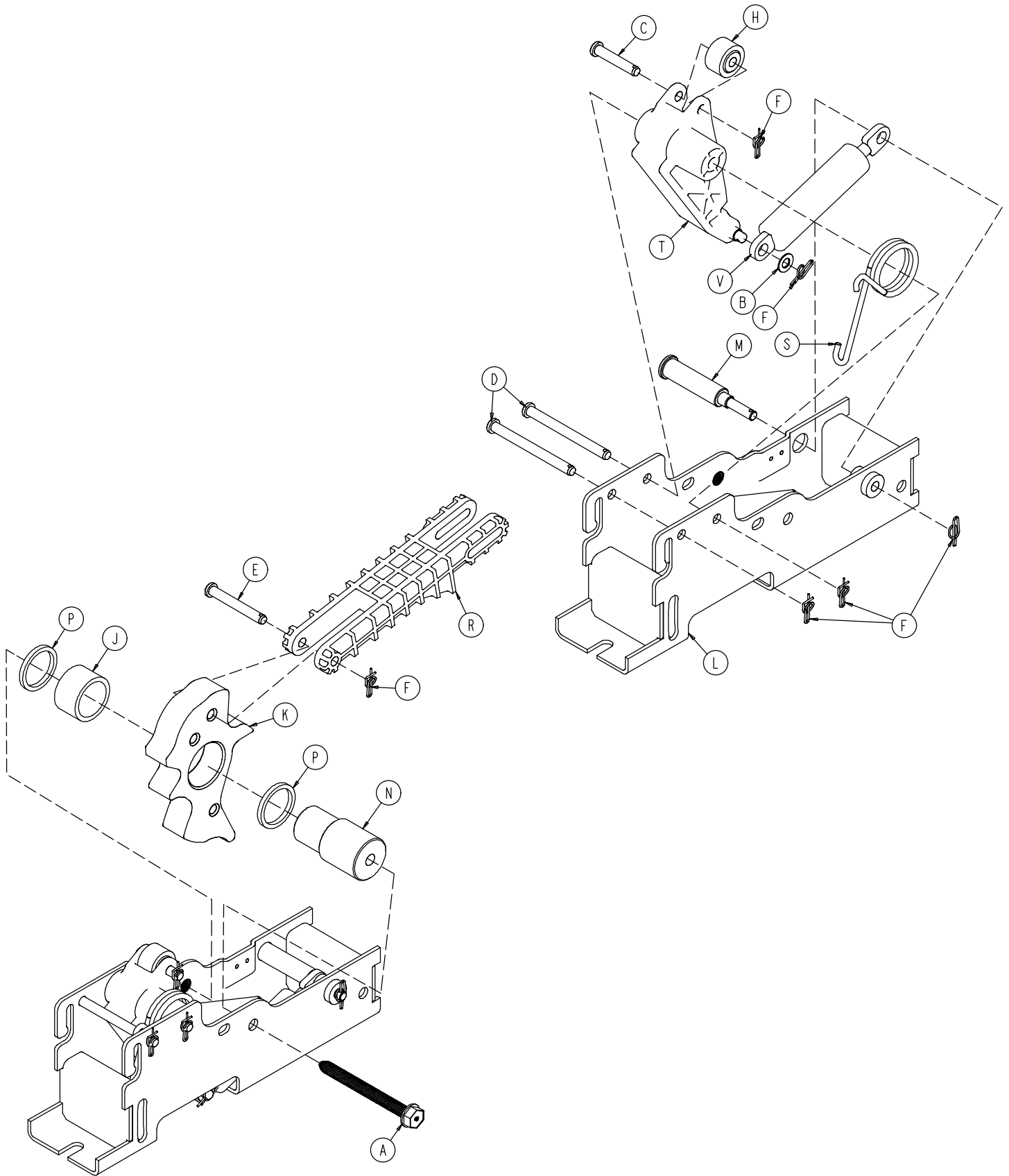


1040-006-101


A	0023-151-000	1
B	0014-004-000	1
C	0026-273-000	1
D	0026-341-000	2
E	0026-330-000	1
F	0027-020-000	6
H	0081-305-000	1
J	0081-400-000	1
K	1040-006-113	1
L	1040-006-121	1
M	1040-006-118	1
N	1040-006-141	1
P	1040-015-008	2
R	1040-006-144	1
S	1210-001-146	1
T	1040-006-163	1
V	1040-006-109	1

1040-015-001

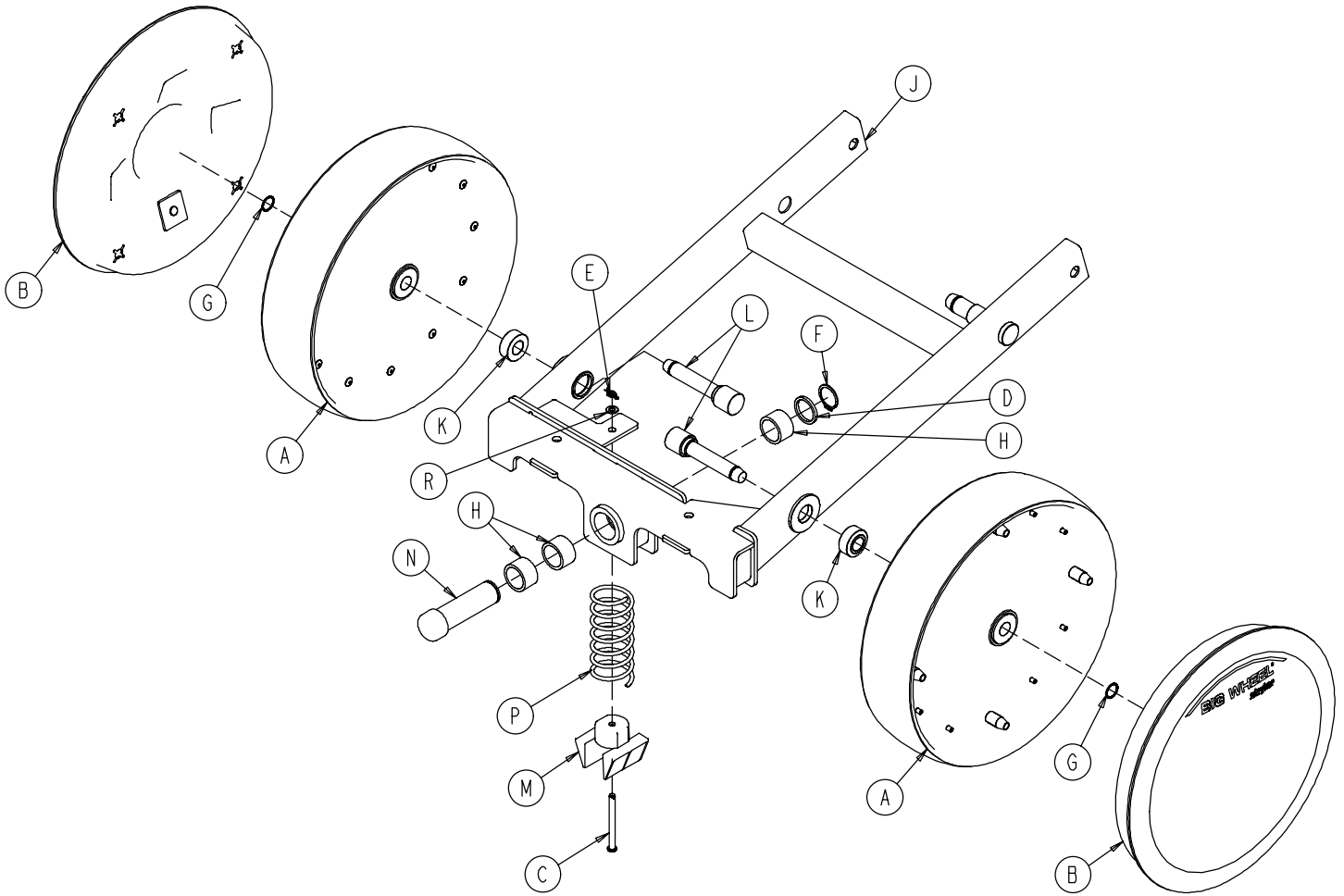


A	1210-301-619	2
B	1040-015-030	2
C	0026-341-000	1
D	1040-015-008	1
E	0027-020-000	1
F	0028-173-000	1
G	0028-330-000	2
H	0081-400-000	3
J	1040-015-010	1
K	1040-015-051	2
L	1040-015-052	2
M	1040-015-053	1
N	1040-015-028	1
P	1040-015-006	1
R	0014-004-000	1

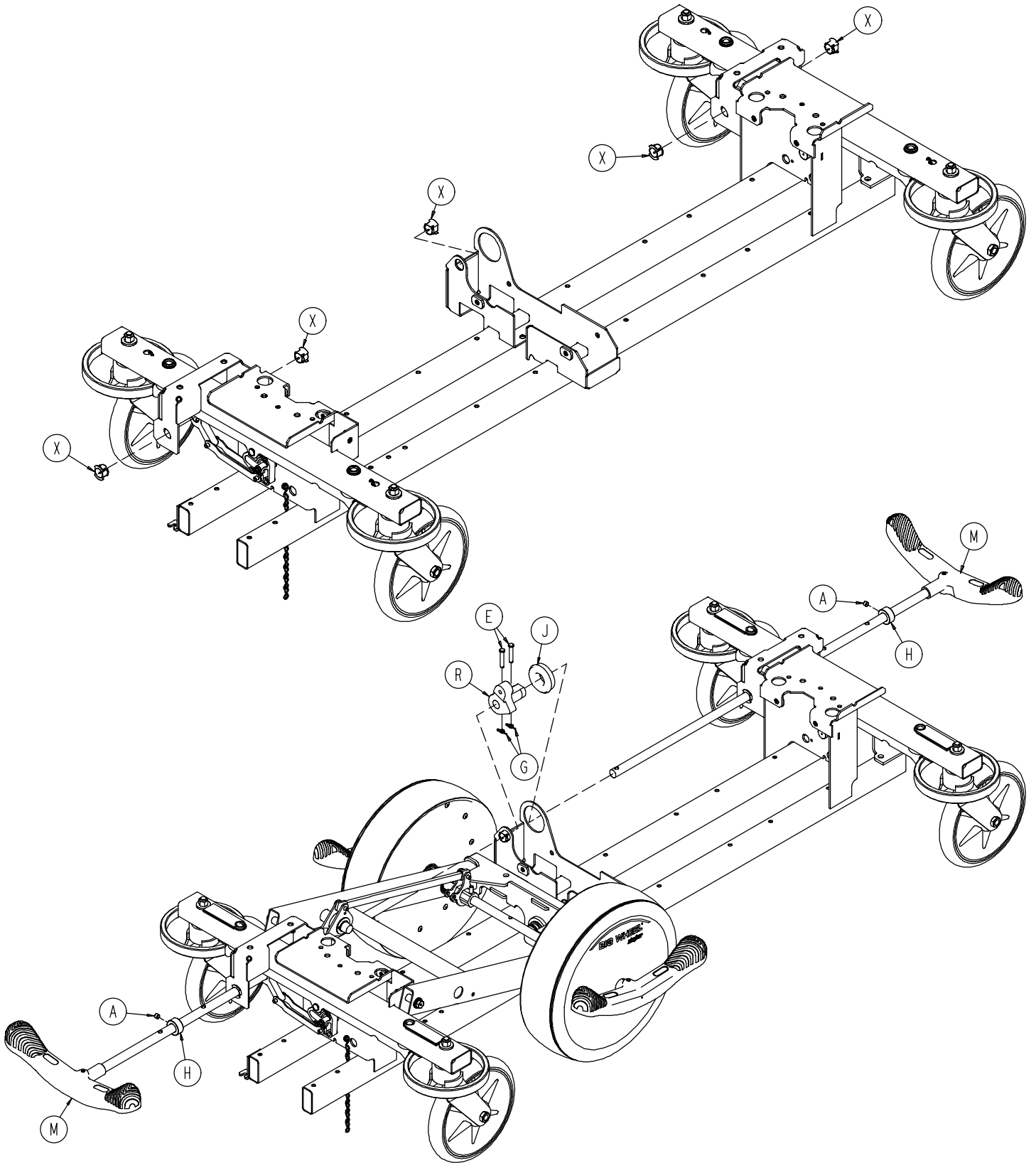


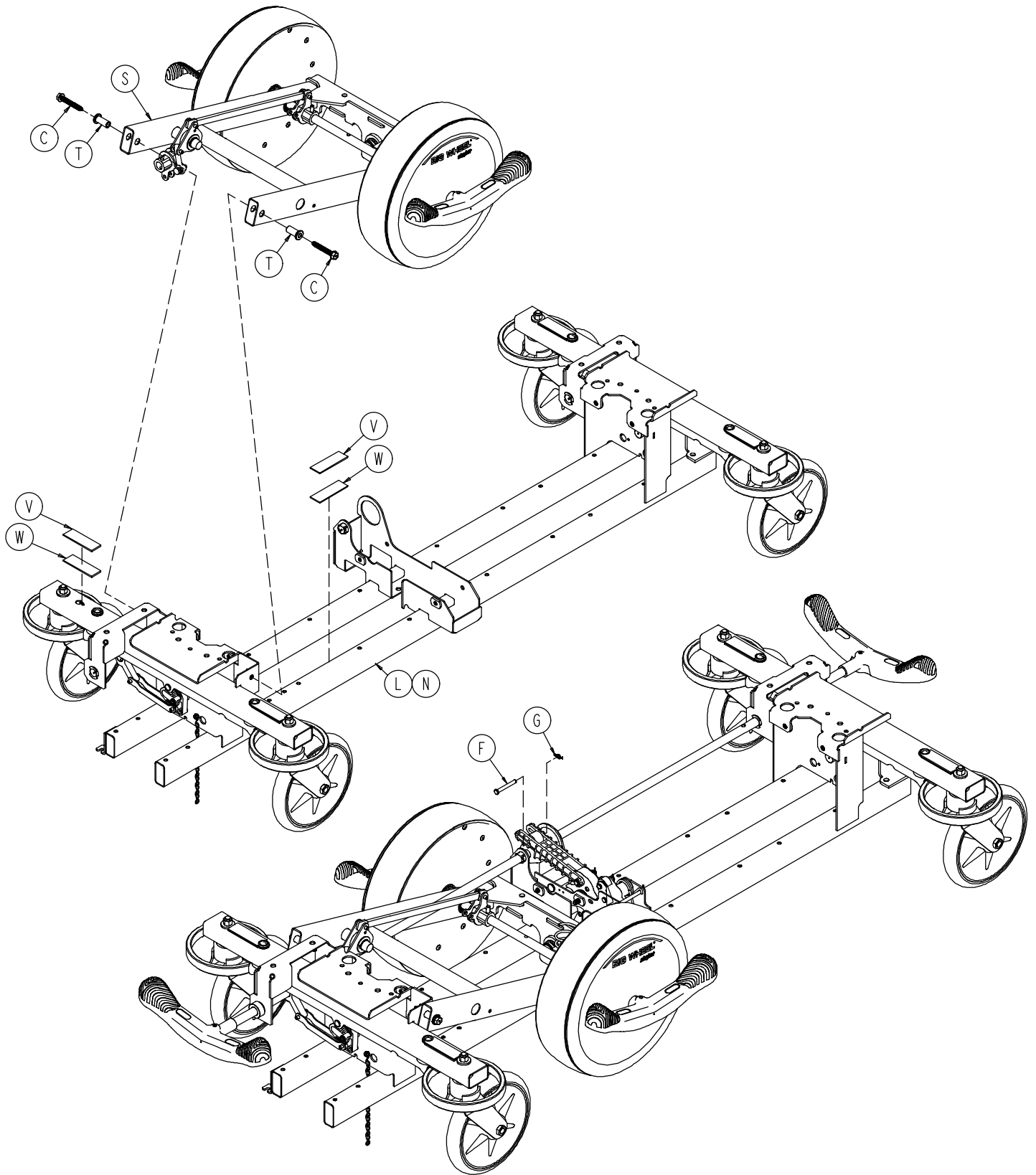

1040-025-005

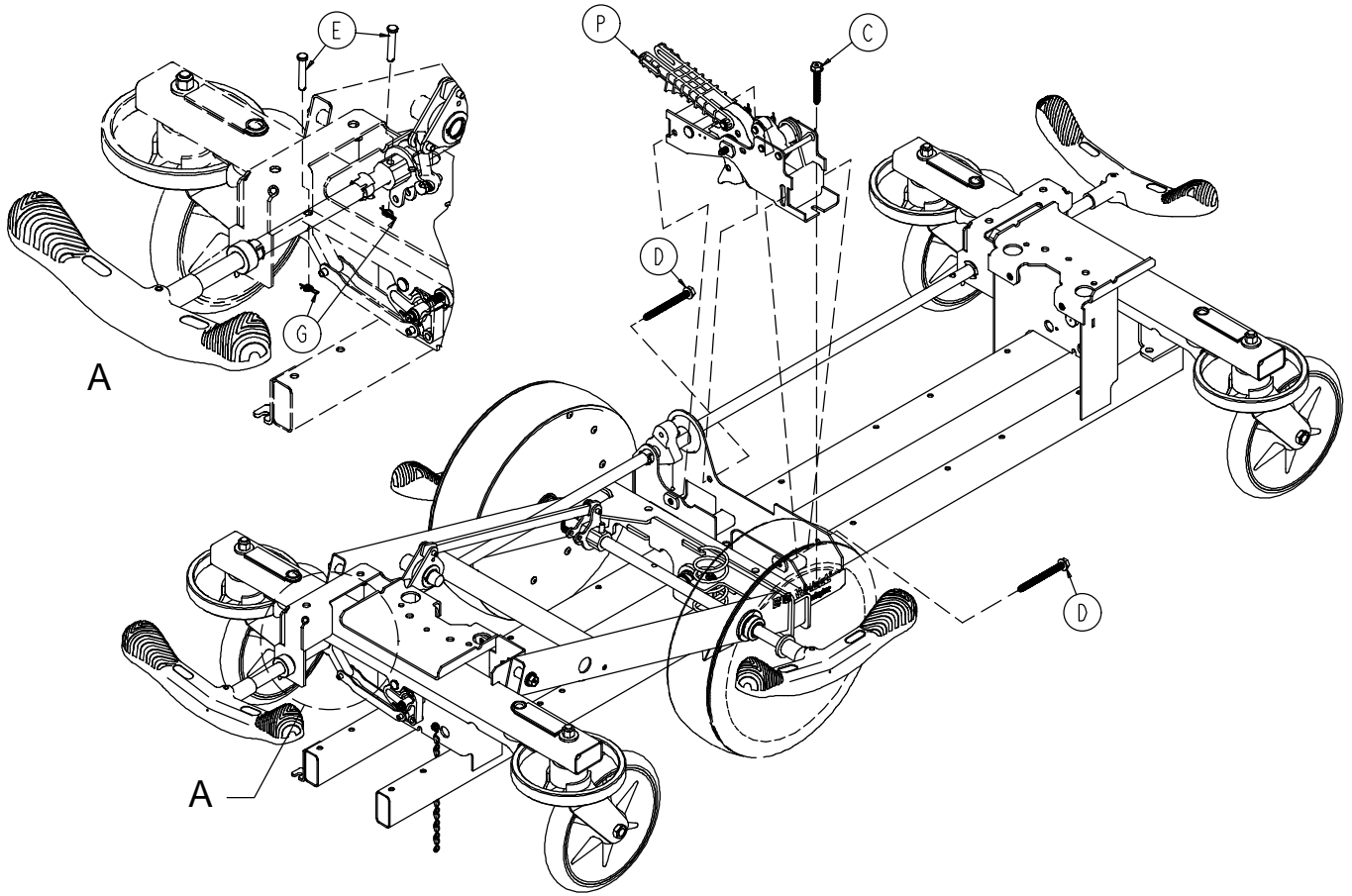


A	0021-166-000	2
C	0023-150-000	3
D	0023-151-000	2
E	0026-273-000	5
F	0026-330-000	1
G	0027-020-000	6
H	0042-020-000	2
J	0081-330-000	1
K	0753-010-012	1
L	1040-003-105	1
M	1040-003-191	2
N	1040-005-200	1
P	1040-006-101	1
R	1040-006-131	1
S	1040-025-001	1
T	1040-015-007	2
V	0029-007-000	5
W	0029-009-000	5
X	1210-201-251	5

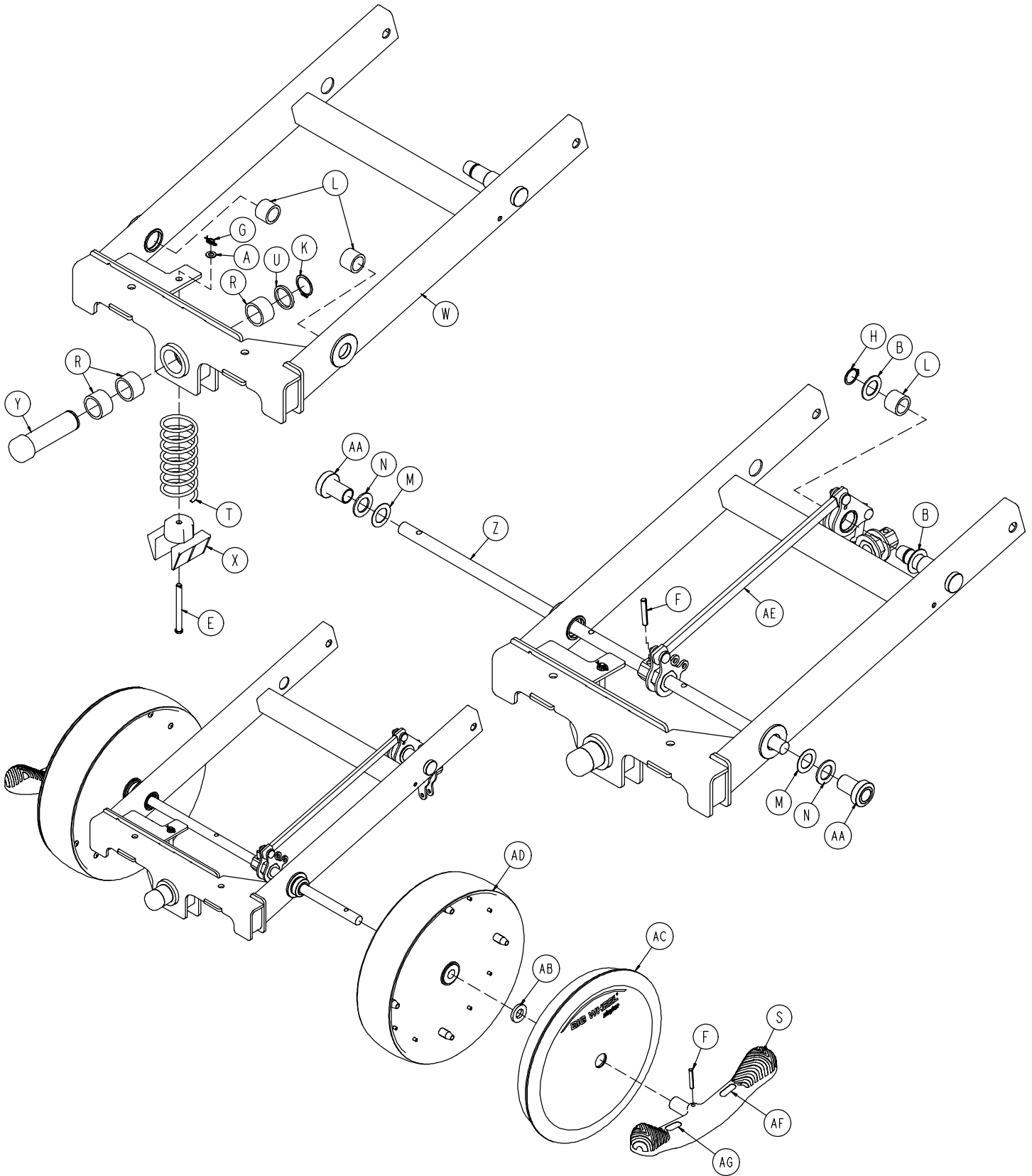

1040-025-001


A	0014-004-000	1
B	0011-148-000	2
E	0026-341-000	1
F	0026-342-000	3
G	0027-020-000	1
H	0028-166-000	1
K	0028-173-000	1
L	0081-011-000	3
M	0011-525-000	2
N	0081-175-000	2
R	0081-400-000	3
S	0753-003-099	2
T	1040-015-006	1
U	1040-015-008	1
W	1040-015-010	1
X	1040-015-053	1
Y	1040-020-028	1
Z	0753-003-119	1
AA	1040-025-014	2
AB	1040-025-017	2
AC	1040-025-030	2
AD	1210-301-619	2
AE	1040-025-010	1
AF	1210-201-335	2
AG	1210-201-336	2

1040-025-010

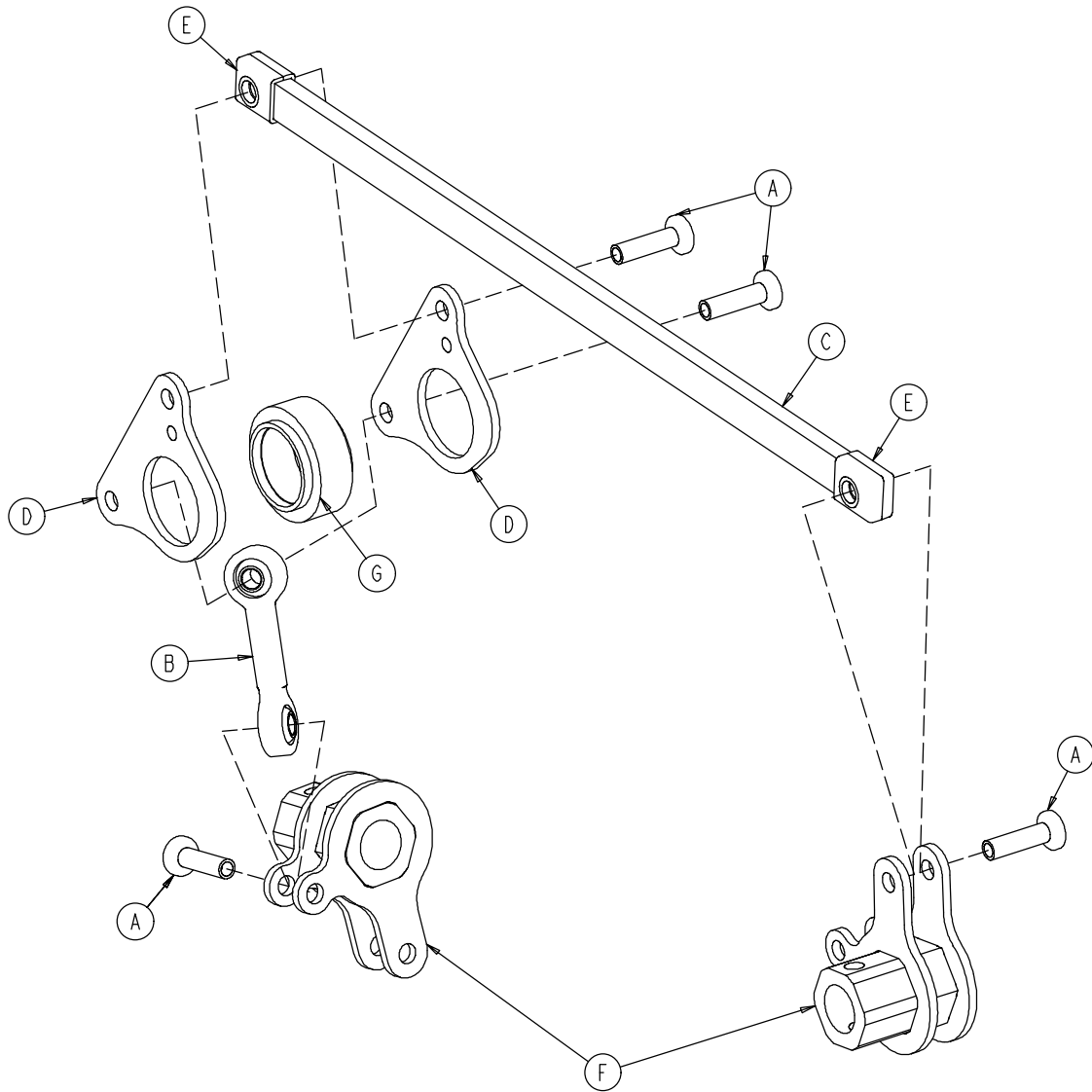

A	0753-003-098	4
B	0753-003-117	1
C	1040-025-011	1
D	1040-025-013	2
E	1040-025-016	2
F	1040-025-020	2
G	1040-025-018	1

1040-015-400

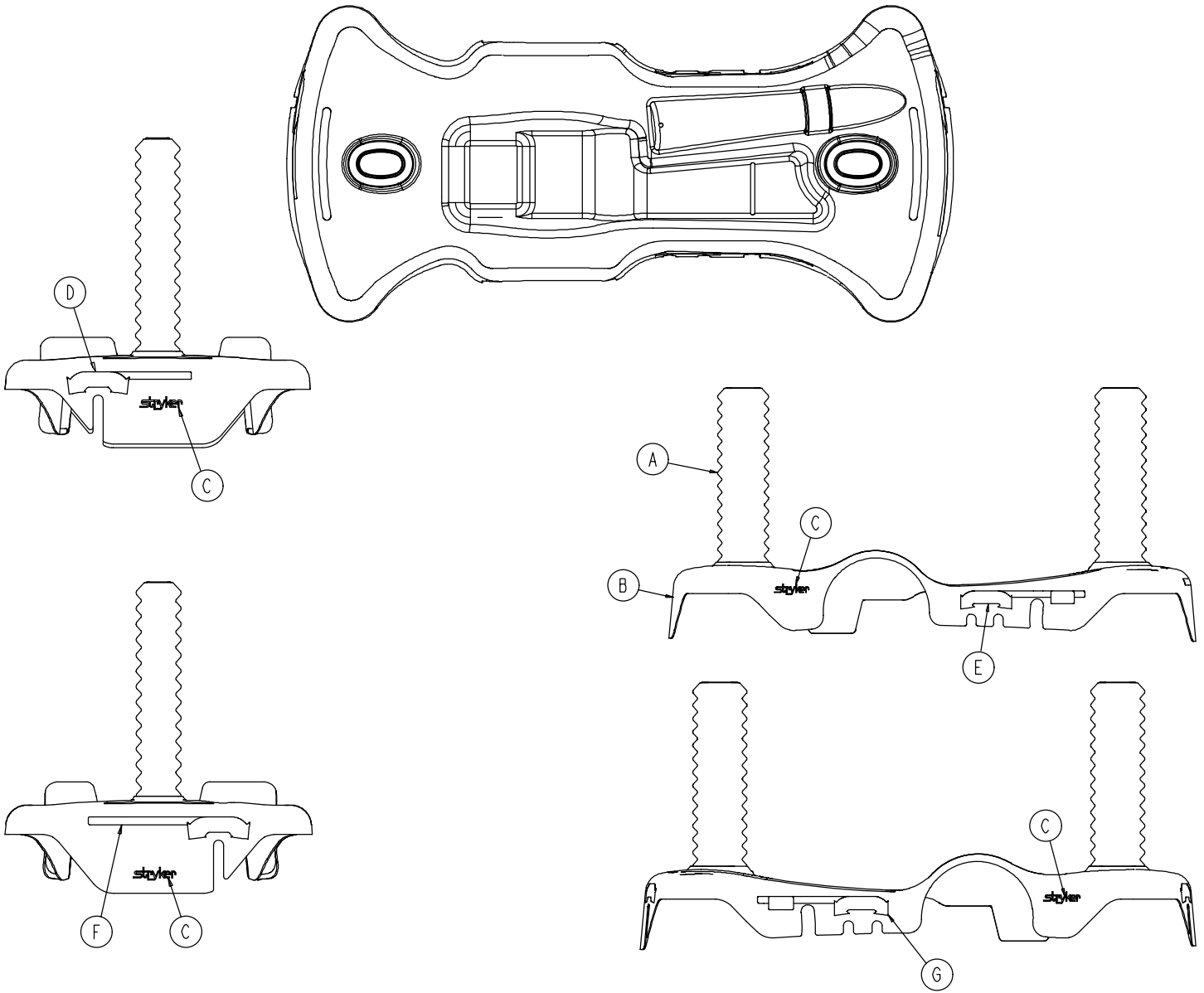


A	1040-010-134	2
B	1040-015-404	1
C	0946-201-060	4

Labels


	D	E	F	G
RED 0753-020-120	0753-010-091	0753-010-055	0753-010-057	0753-010-054
PURPLE 0753-020-121	0753-010-191	0753-010-155	0753-010-210	0753-010-154
GREEN 0753-020-122	0753-010-291	0753-010-255	0753-010-310	0753-010-254
GRAY 0753-020-123	0753-010-391	0753-010-355	0753-010-410	0753-010-354
TEAL 0753-020-124	0753-010-491	0753-010-455	0753-010-510	0753-010-454
PINK 0753-020-125	0753-010-591	0753-010-555	0753-010-610	0753-010-554
BLUE 0753-020-126	0753-010-691	0753-010-655	0753-010-710	0753-010-654

0753-002-080

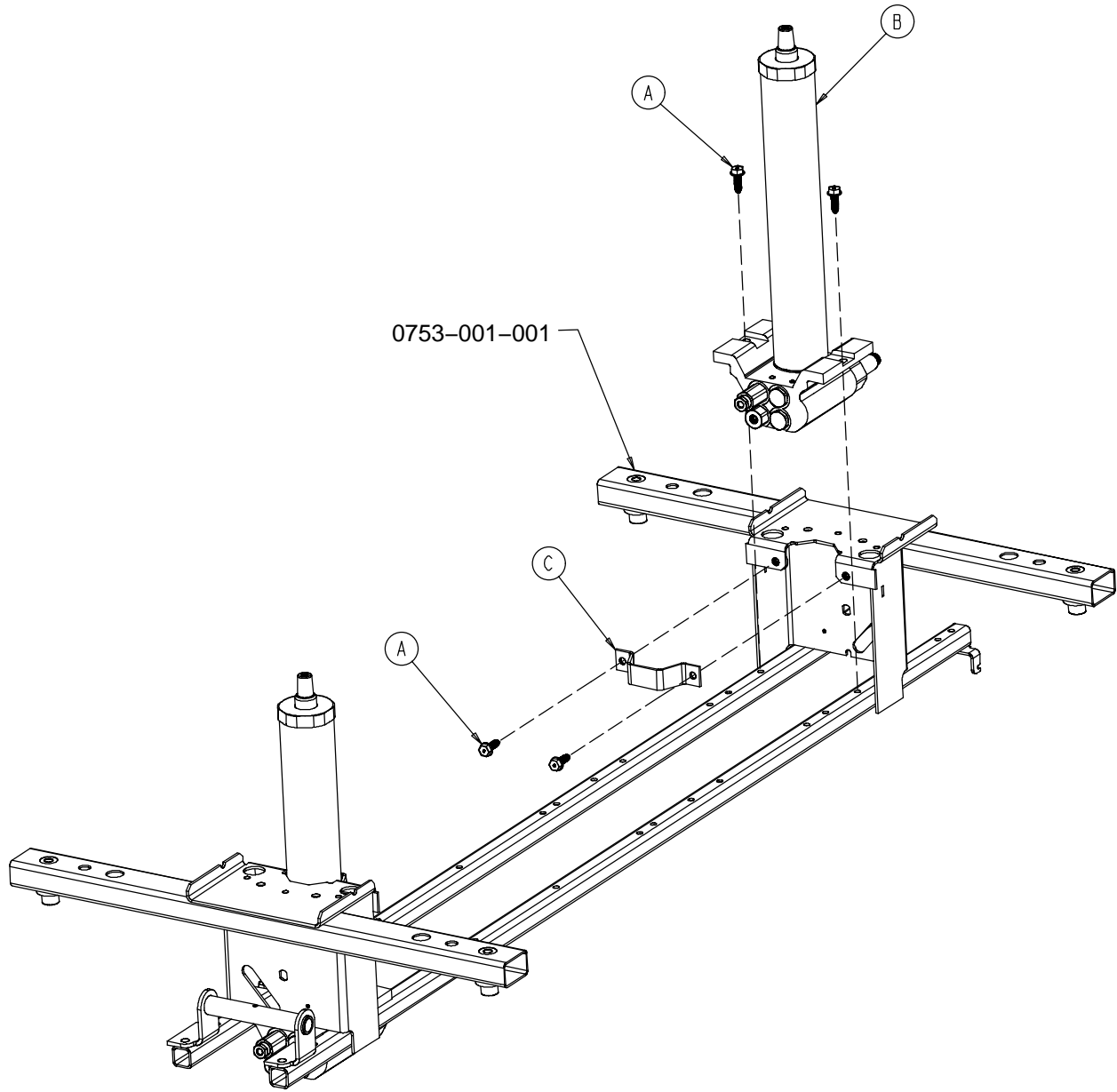



A	0023-288-000	8
B	0753-002-070	2
C	0753-010-007	2

0753-002-025

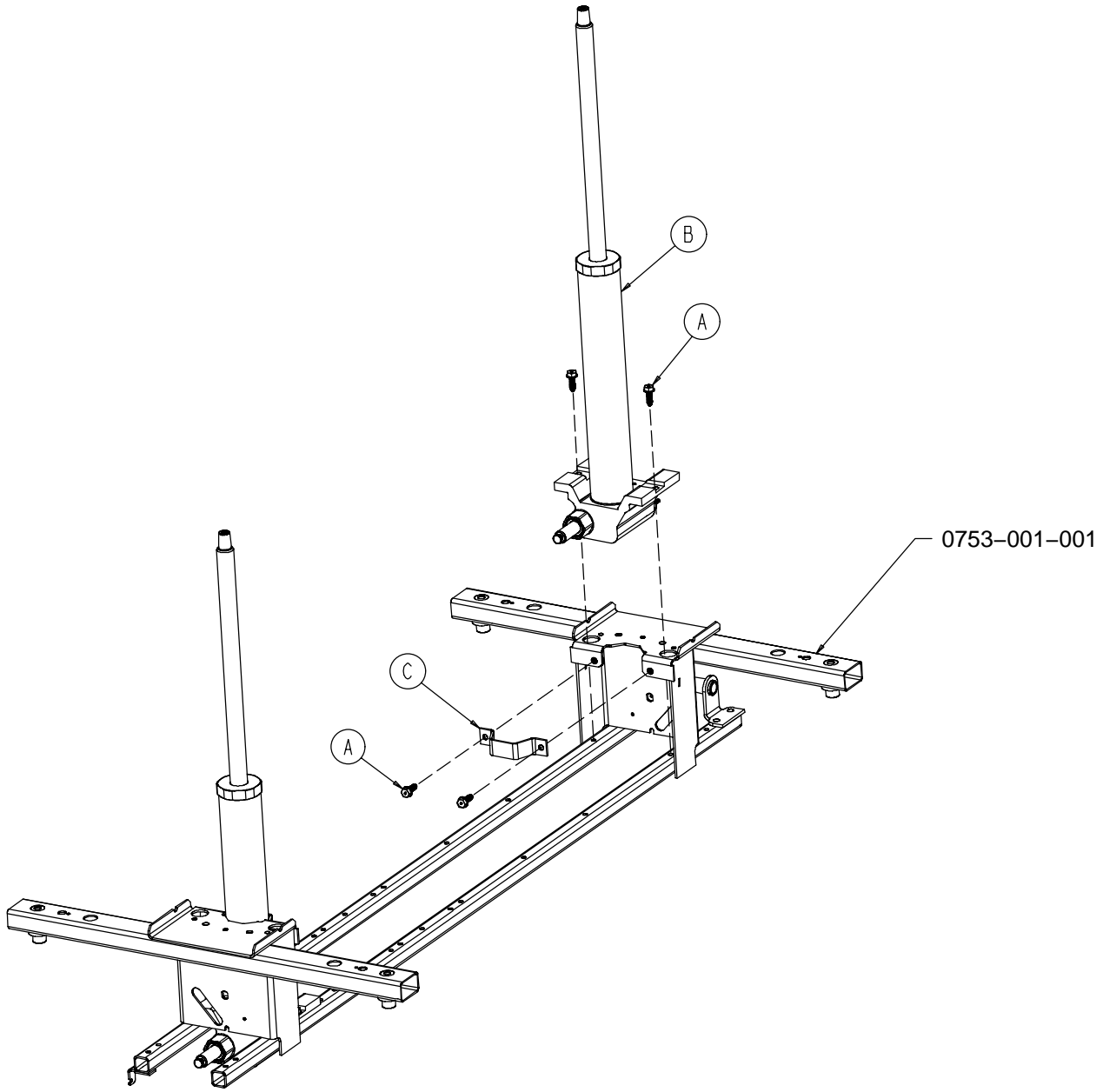


A	0023-288-000	8
B	0753-002-001	2
C	0753-010-007	2

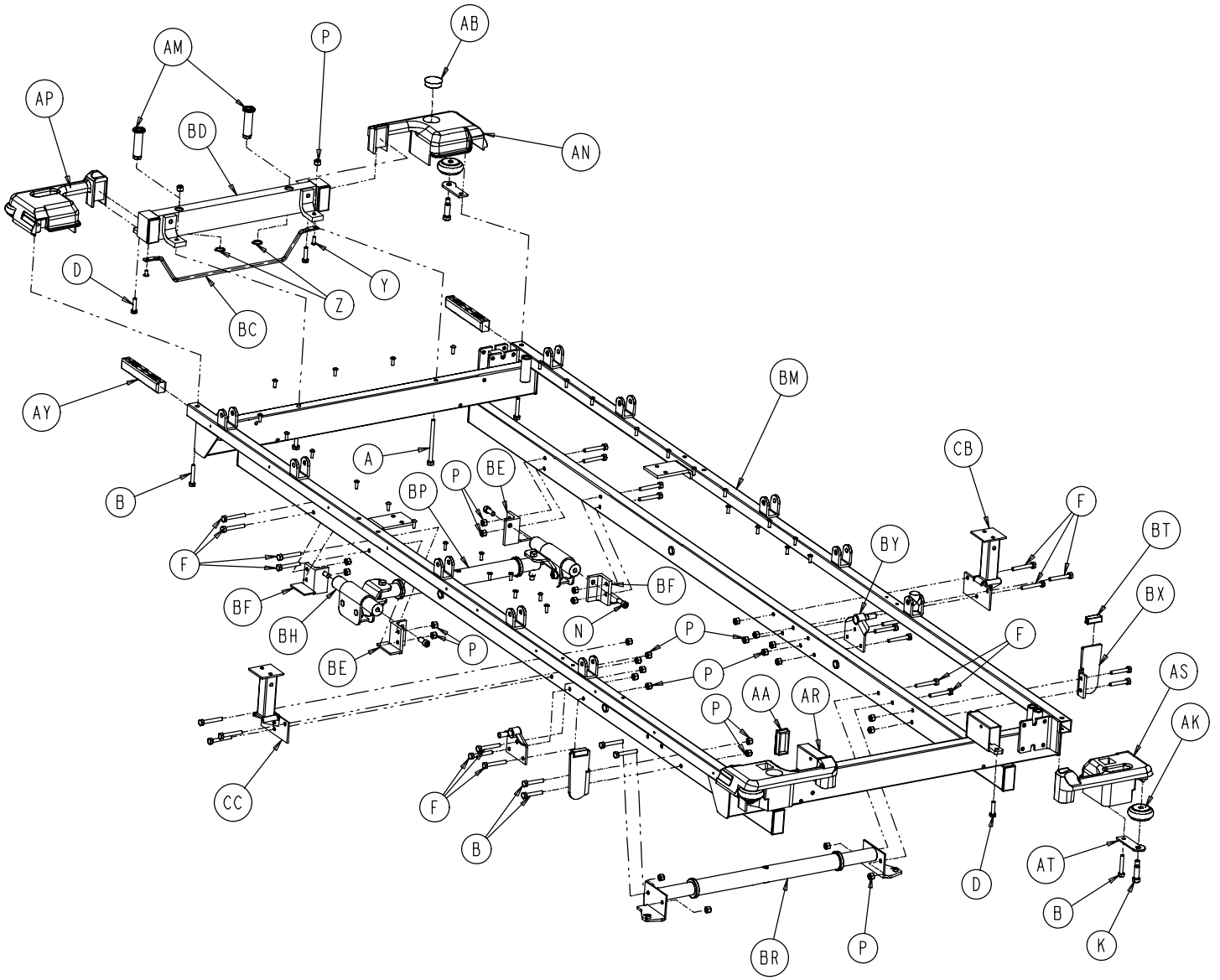

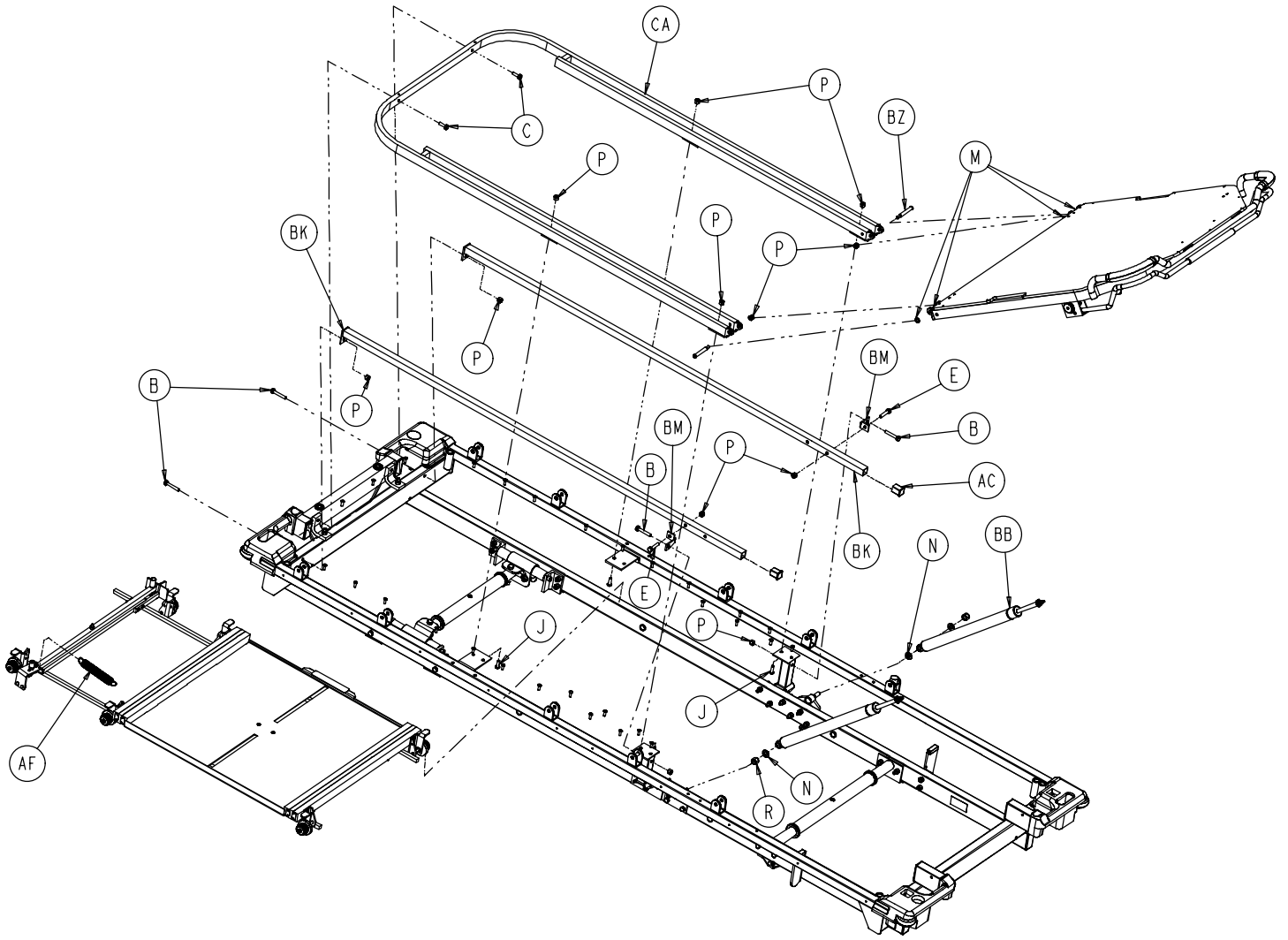


1020-191-010

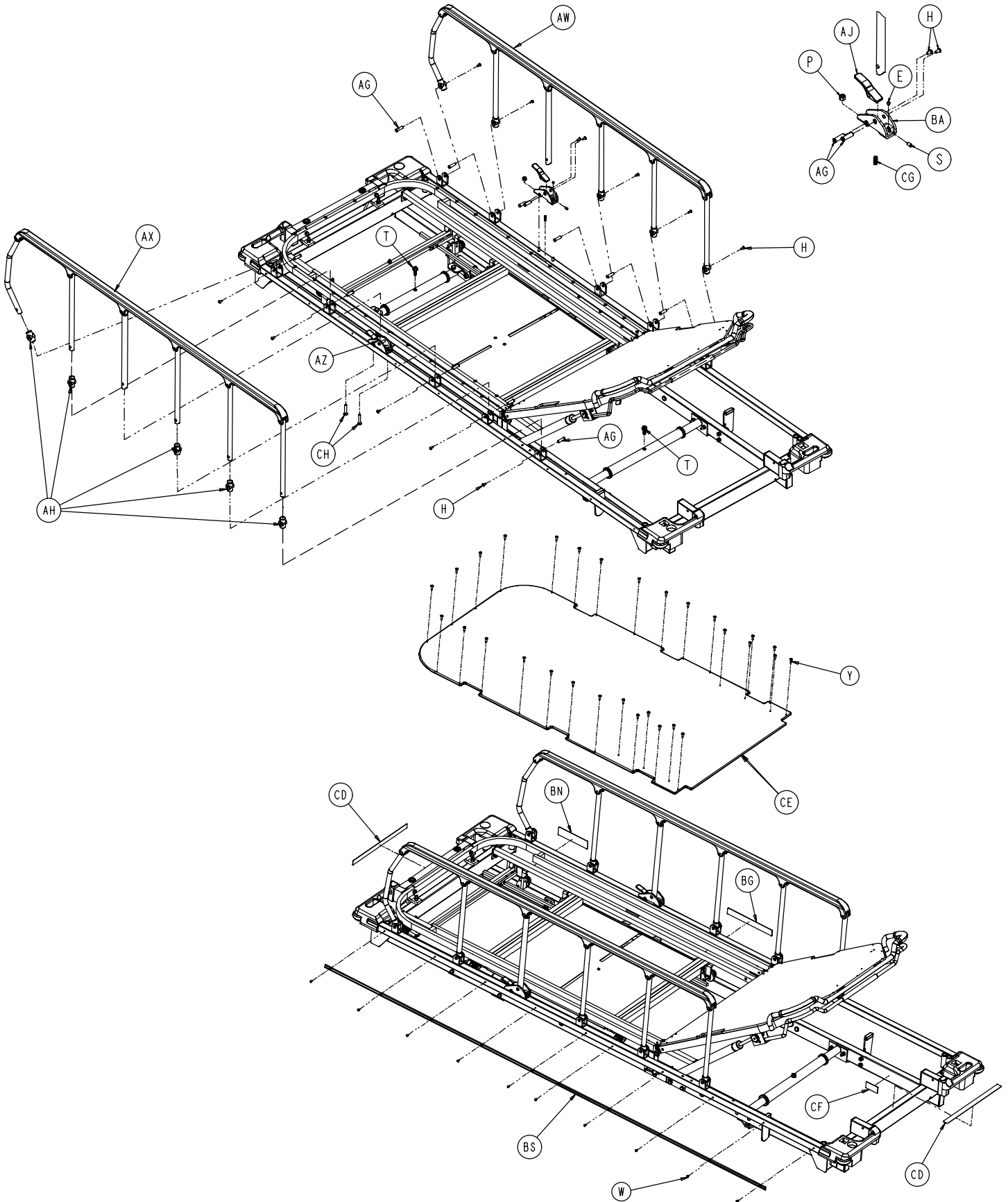

1020-191-010


A	0003-035-000	2	AT	1010-001-017	4
B	0003-047-000	12	AW	1010-026-017	1
C	0003-050-000	2	AX	1010-026-019	1
D	0003-054-000	4	AY	1010-026-069	2
E	0003-074-000	2	AZ	1010-026-080	1
F	0003-078-000	24	BA	1010-026-081	1
G	0004-021-000	4	BB	1010-031-078	2
H	0004-136-000	14	BC	1010-032-092	1
J	0004-161-000	4	BD	1020-001-040	1
K	0008-024-000	4	BE	1020-001-042	2
M	0014-003-000	4	BF	1020-001-043	2
N	0014-021-000	4	BG	1020-001-053	1
P	0016-028-000	44	BH	1020-001-059	2
R	0016-035-000	2	BJ	1020-001-061	1
S	0021-104-000	2	BK	1020-022-023	2
T	0023-288-000	2	BM	1020-022-026	2
W	0025-038-000	20	BN	1020-091-028	1
Y	0025-171-000	32	BP	1020-024-002	1
Z	0028-023-000	2	BR	1020-024-003	1
AA	0037-049-000	8	BS	1020-026-021	2
AB	0037-059-000	1	BT	1020-031-009	2
AC	0037-063-000	2	BX	1020-031-019	2
AD	0037-074-000	2	BY	1020-031-025	2
AF	0038-253-000	1	BZ	1020-032-019	2
AG	0721-026-066	14	CA	1020-032-025	1
AH	0721-026-069	10	CB	1020-032-031	1
AJ	0721-026-074	2	CC	1020-032-032	1
AK	0721-032-049	4	CD	1020-091-052	2
AM	0721-040-026	2	CE	1020-032-026	1
AN	1010-001-013	1	CF	1210-800-008	1
AP	1010-001-014	1	CG	0038-220-000	2
AR	1010-001-015	1	CH	0001-096-000	4
AS	1010-001-016	1			

1010-026-017, 1010-026-019

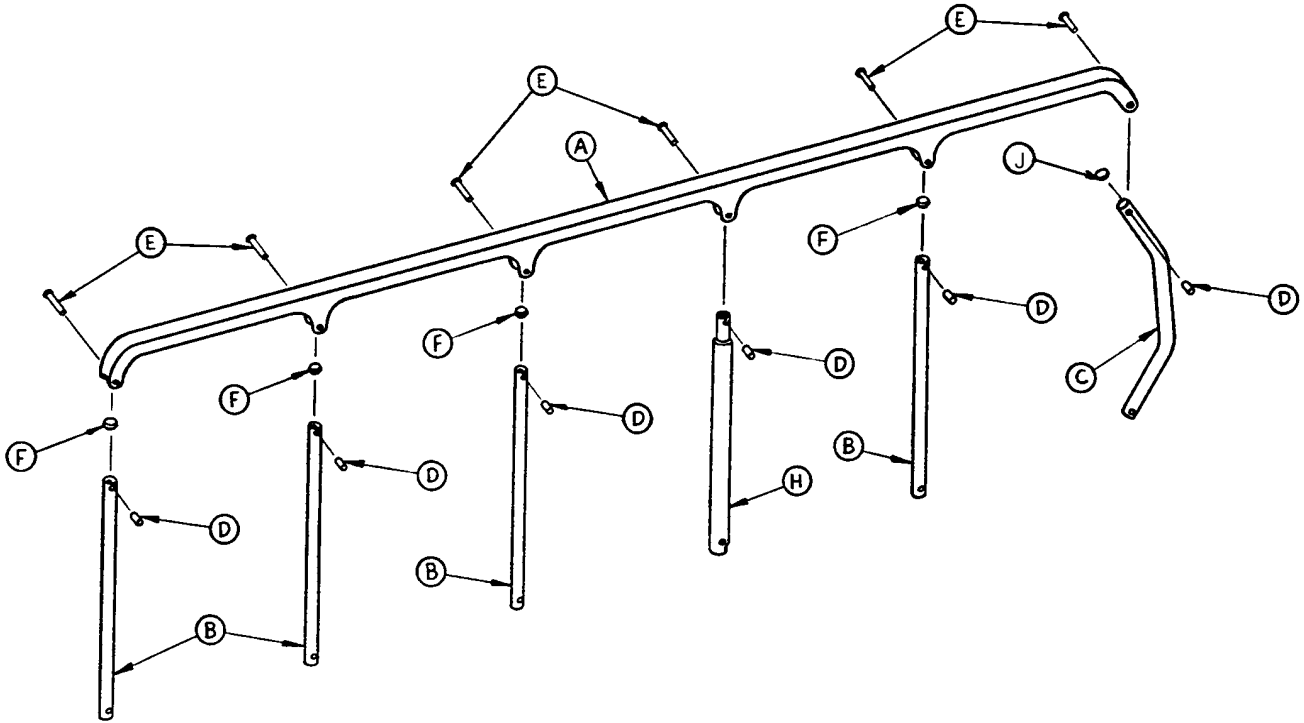


A	1010-026-015	1
B	1010-026-083	4
C	1010-026-084	1
D	1010-026-082	6
E	0025-106-000	6
F	1010-026-010	4
H	1010-026-085	1
J	1010-026-012	1

1020-024-002

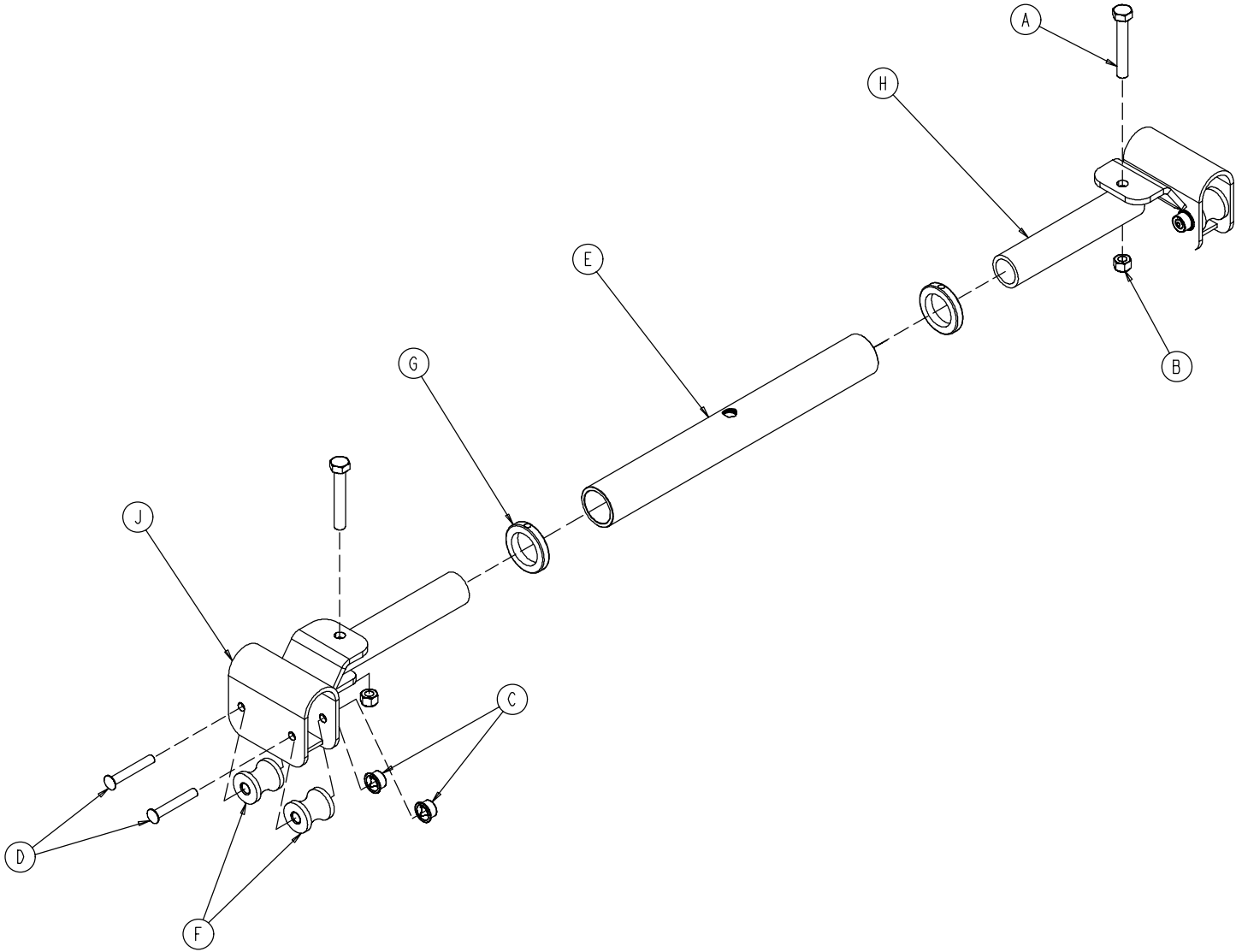


A	0003-040-000	2
B	0016-028-000	2
C	0017-003-000	4
D	0025-053-000	4
E	0753-010-029	1
F	0938-001-292	4
G	0938-001-401	2
H	1010-032-064	2
J	1020-001-048	2

1020-024-003

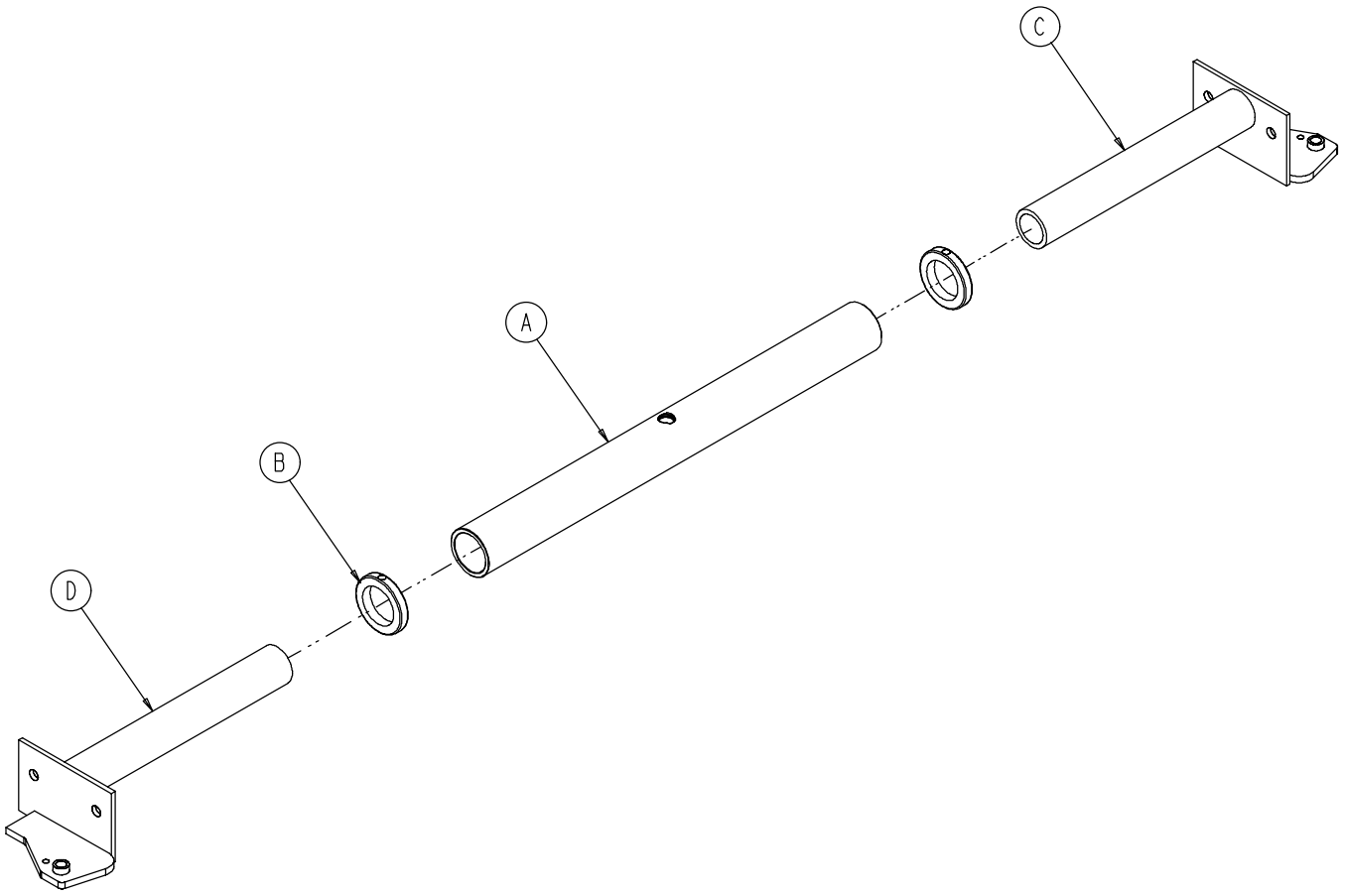


A	0753-010-03	1
B	0938-001-401	2
C	1020-024-004	1
D	1020-024-005	1

1020-032-059

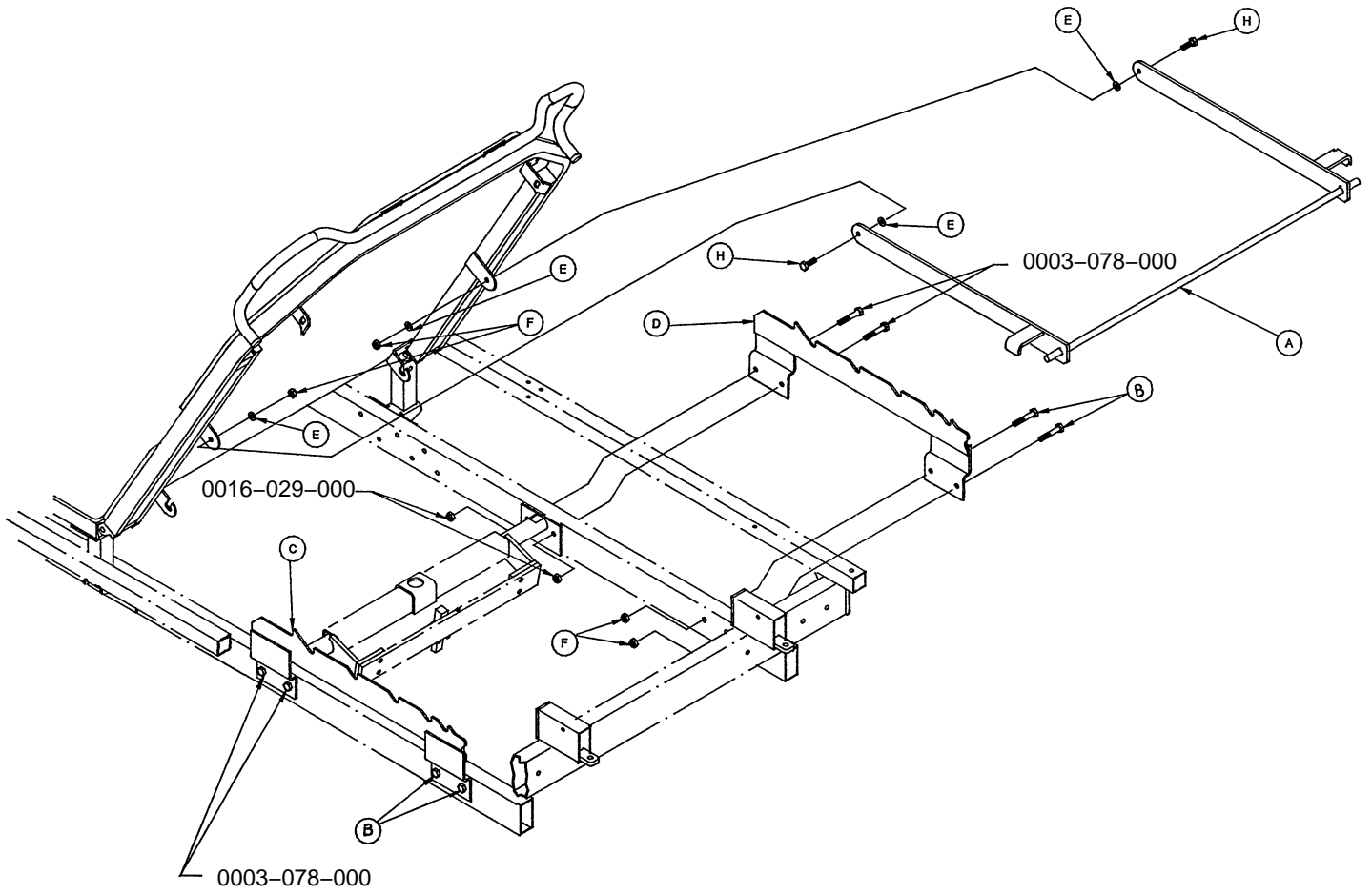


A	1020-032-054	1
B	0003-078-000	4
C	1020-032-056	1
D	1020-032-055	1
E	0014-002-000	4
F	0016-028-000	6
H	0003-050-000	2

1020-032-058

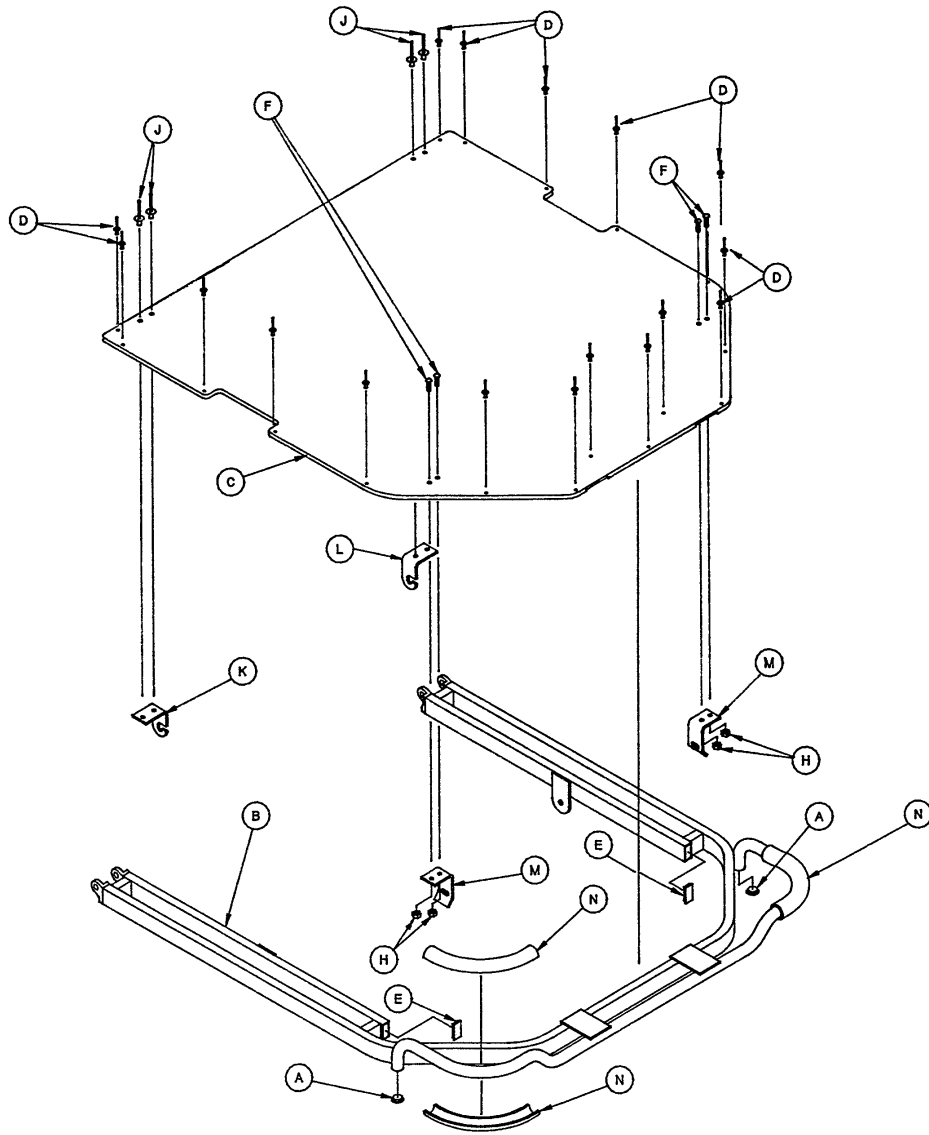


A	0037-065-000	2
B	1020-032-057	1
C	1020-031-039	1
D	0025-099-000	17
E	1010-032-066	2
F	0004-149-000	4
H	0016-003-000	4
J	0025-069-000	4
K	1020-023-012	1
L	1020-023-013	1
M	1020-023-014	2
N	1020-001-065	4

1020-031-011

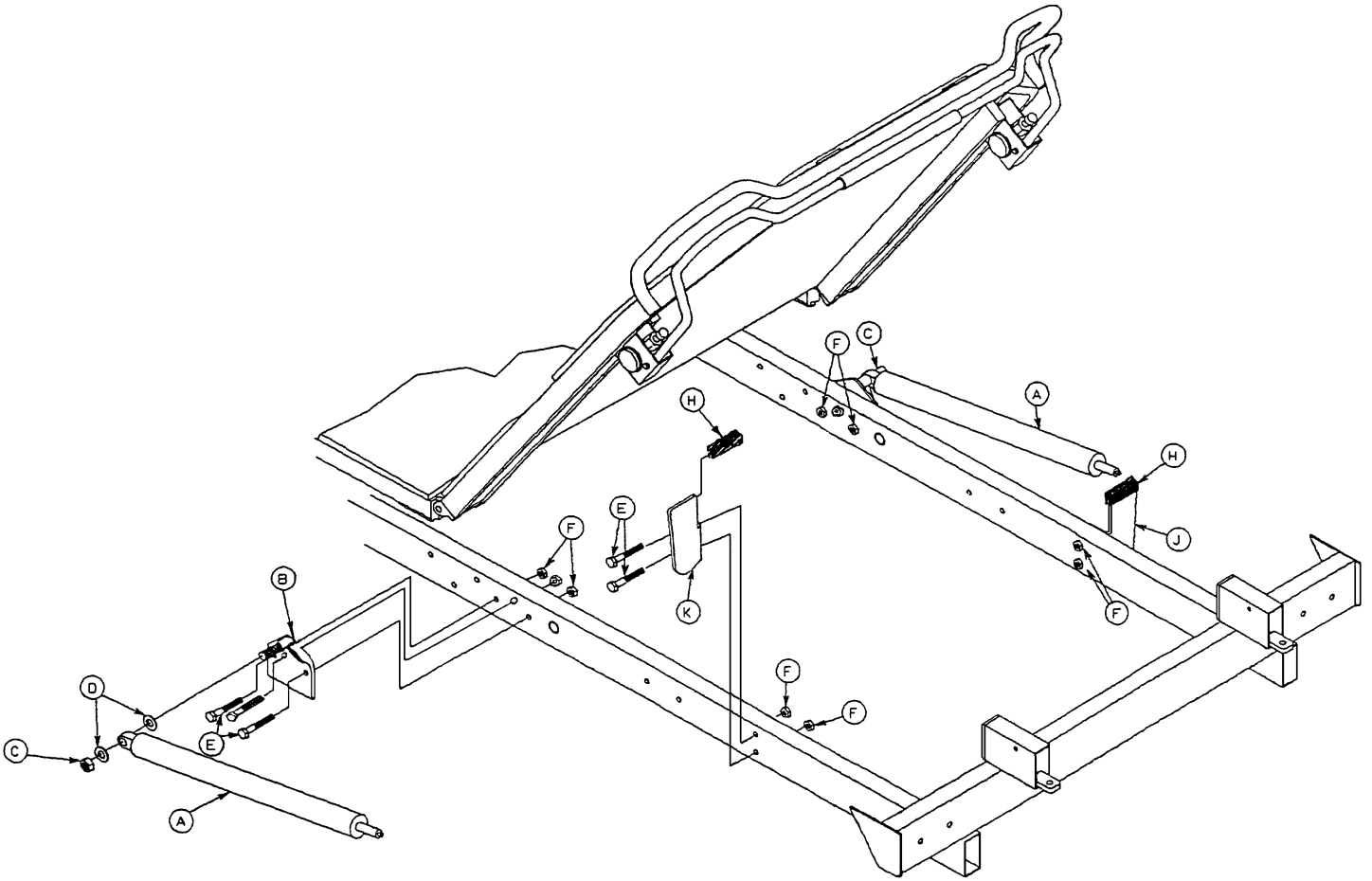


A	1010-031-078	2
B	1020-031-025	2
C	0016-035-000	2
D	0014-029-000	4
E	0003-078-000	10
F	0016-028-000	10
H	1020-031-038	2
J	1020-031-017	1
K	1020-031-019	1

1020-031-010

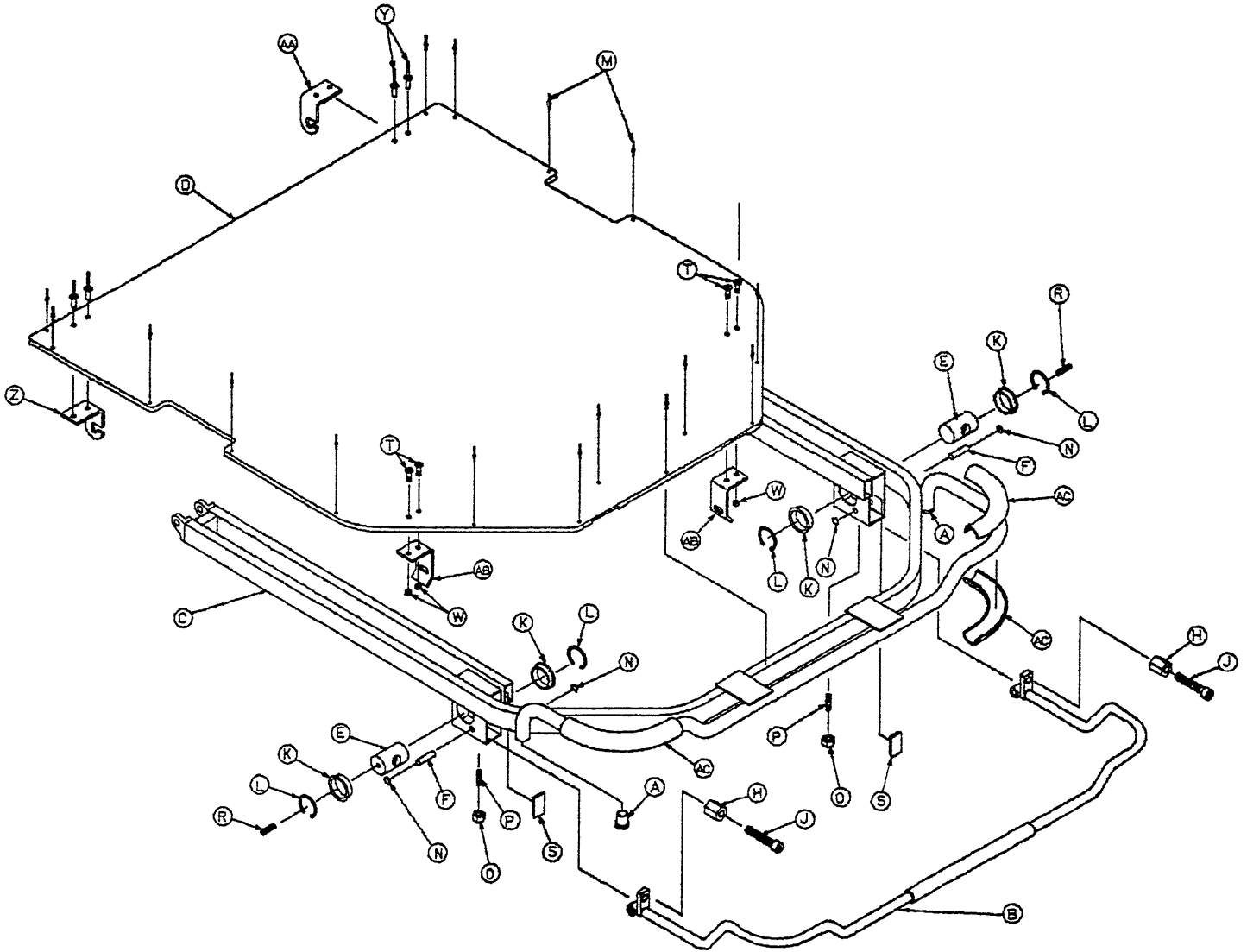


A	0037-065-000	2	O	0015-037-000	2
B	1020-031-020	1	P	0021-104-000	2
C	1020-031-015	1	R	0021-119-000	2
D	1020-031-039	1	S	1010-032-066	2
E	1510-031-028	2	T	0004-149-000	4
F	1510-031-037	2	W	0016-003-000	4
H	0015-050-000	2	Y	0025-069-000	4
J	0021-126-000	2	Z	1020-023-012	1
K	0946-035-025	4	AA	1020-023-013	1
L	0028-076-000	4	AB	1020-023-014	2
M	0025-099-000	17	AC	1020-001-065	4
N	0028-008-000	4			

1020-151-001

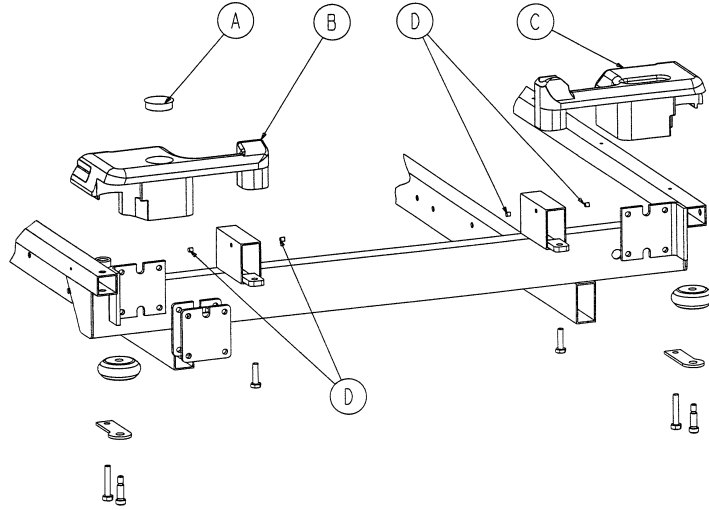


A	0037-059-000	1
B	1010-001-013	1
C	1010-001-014	1
D	0037-074-000	4

1020-151-004

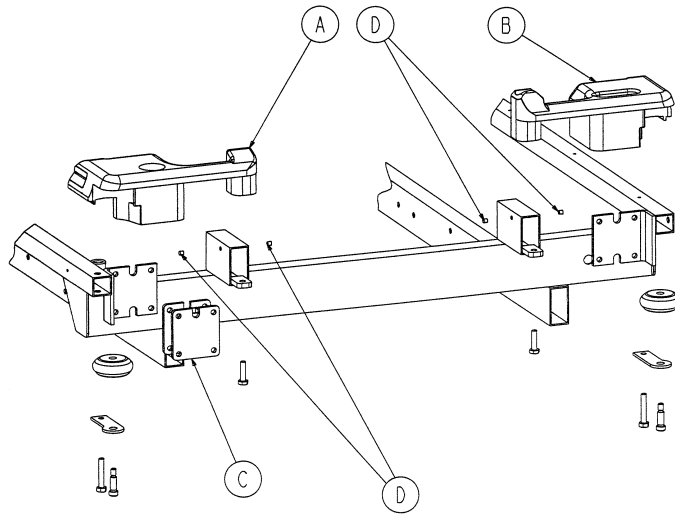


A	1010-001-013	1
B	1010-001-014	1
C	1020-054-067	1
D	0037-074-000	4

1020-151-003

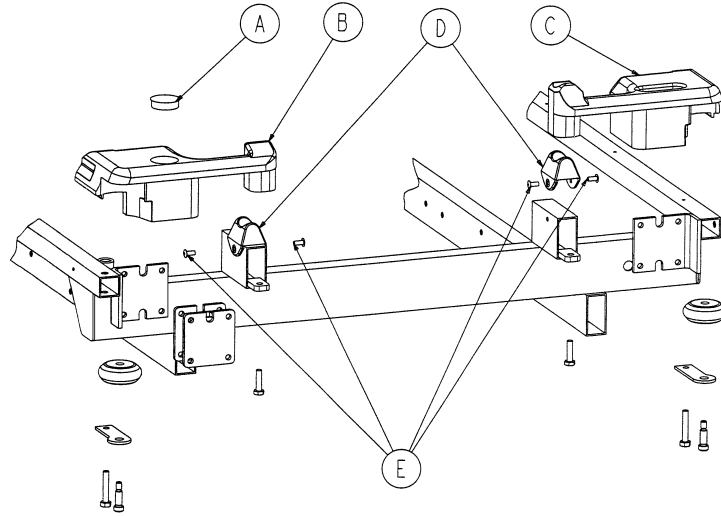


A	0037-059-000	1
B	1010-001-015	1
C	1010-001-016	1
D	1010-001-035	2
E	0025-079-000	4

1020-151-002

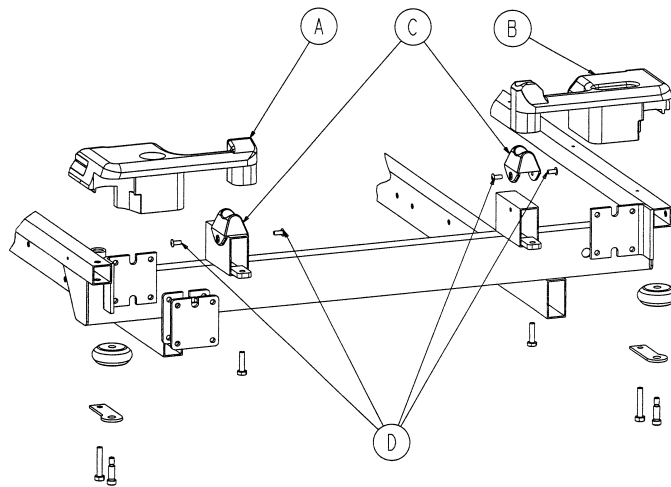


A	1010-001-015	1
B	1010-001-016	1
C	1010-001-035	2
D	0025-079-000	4

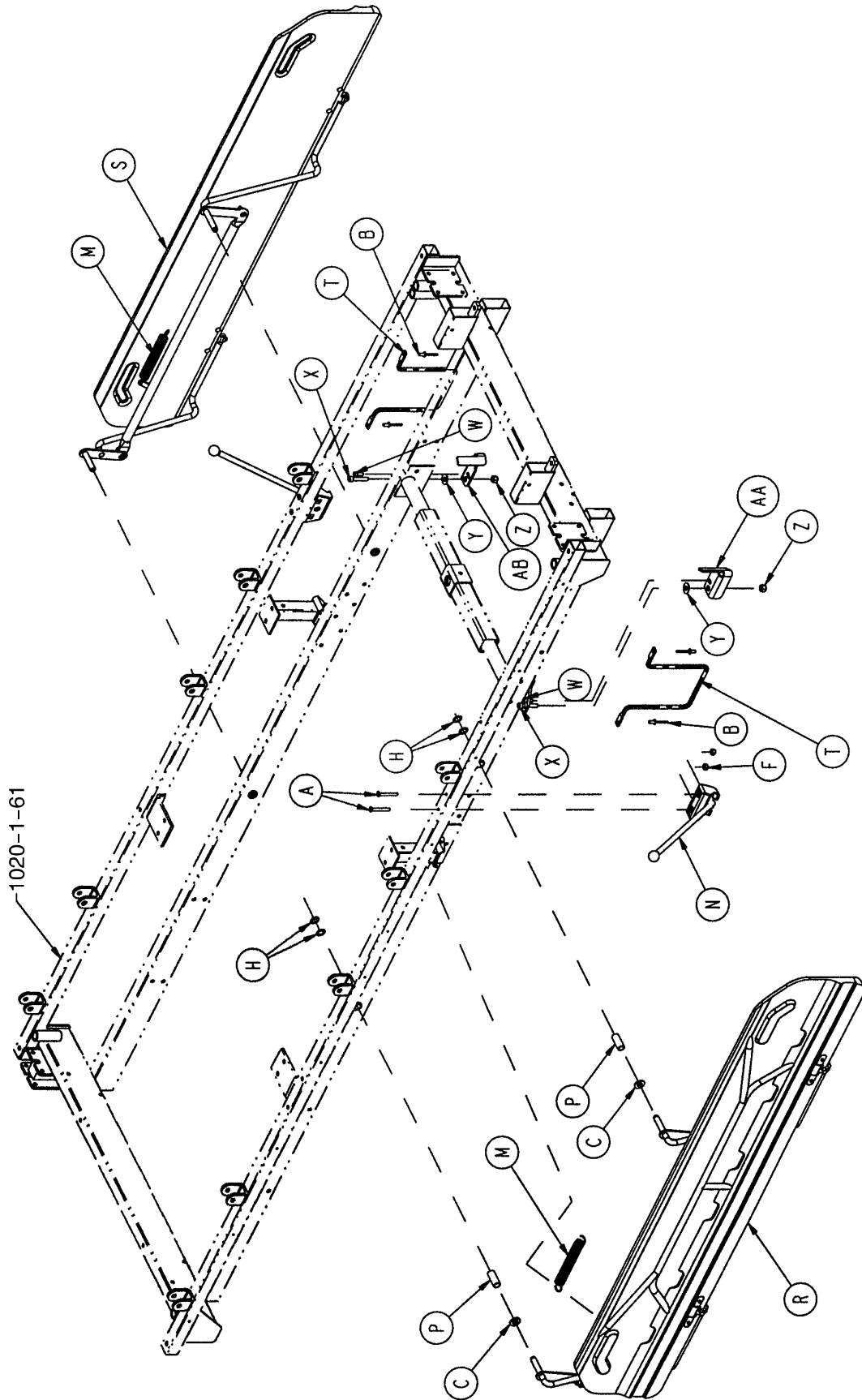

1020-024-020


A	0002-100-000	4
B	0025-079-000	4
C	0014-021-000	4
F	0016-003-000	4
H	0028-093-000	8
M	0038-252-000	2
N	1010-091-030	2
P	1020-001-029	4
R	1020-024-010	1
S	1020-024-011	1
T	1020-024-121	2
W	0004-032-000	2
X	0004-142-000	2
Y	0011-179-000	2
Z	0016-028-000	2
AA	1020-024-034	1
AB	1020-024-035	1

1020-024-010

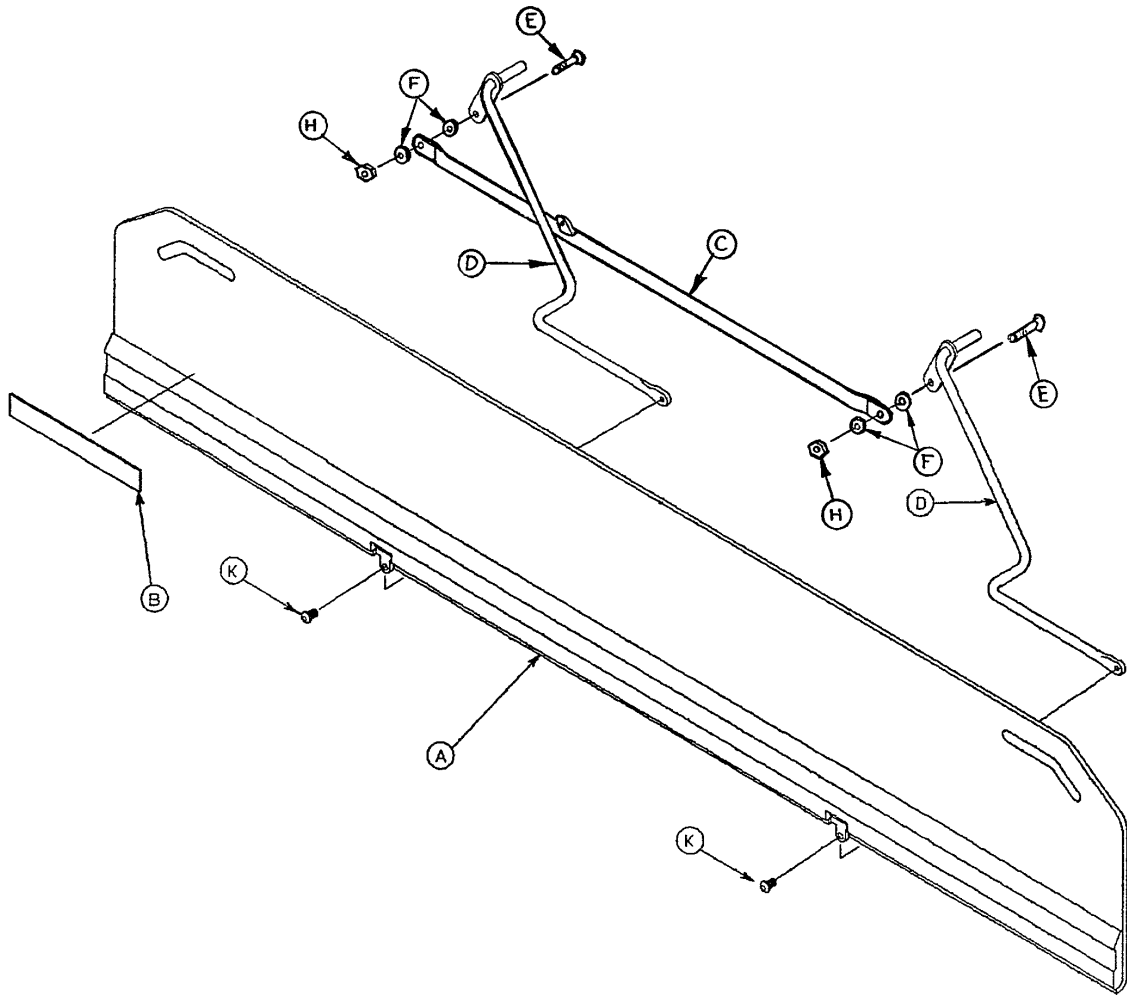


A	1020-032-100	1
B	0721-024-046	1
C	1020-024-038	1
D	1020-024-012	2
E	0001-025-000	2
F	0011-224-000	4
H	0016-028-000	2
K	0004-156-000	2

1020-024-011

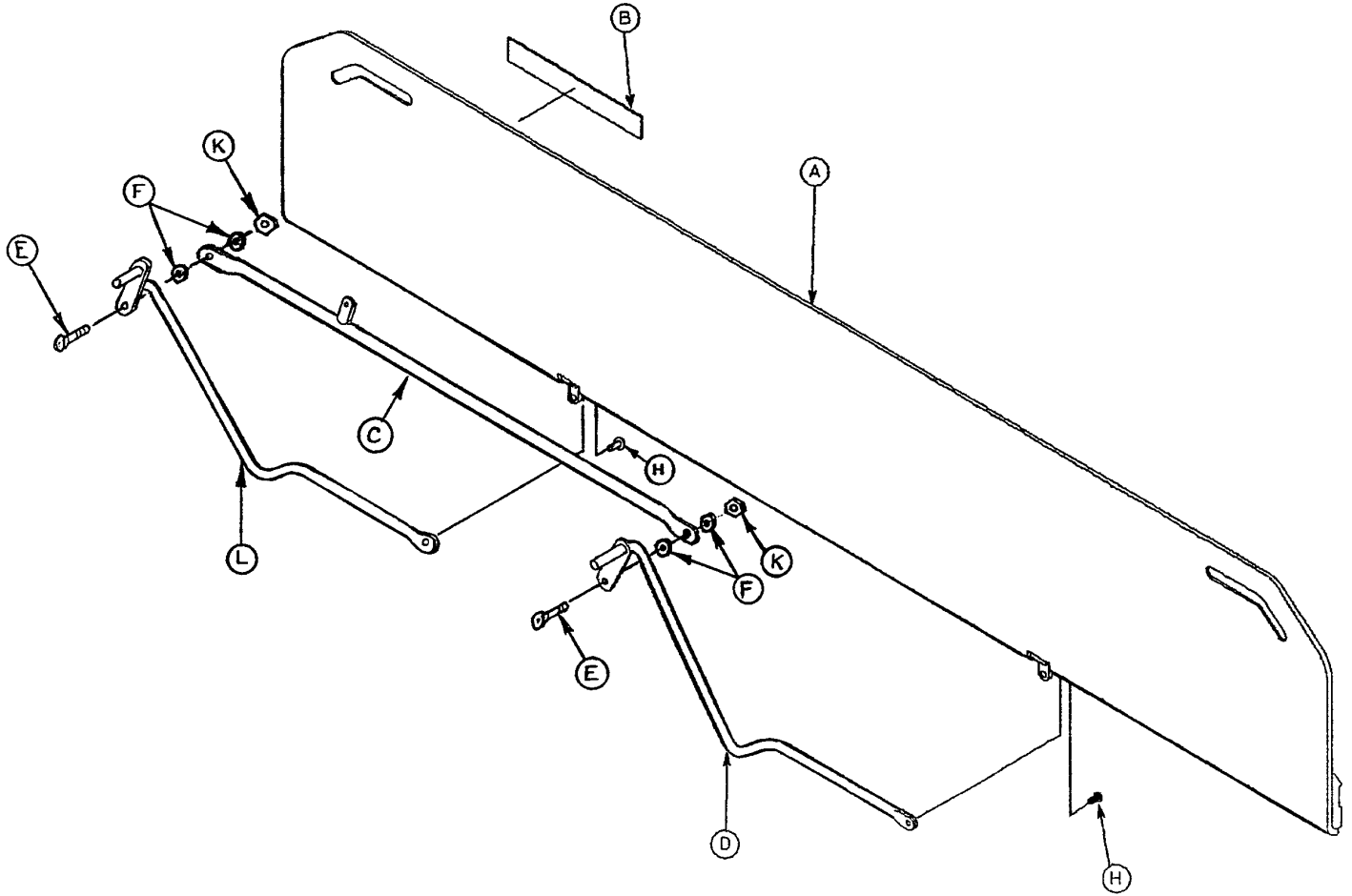


A	1020-032-100	1
B	0721-024-046	1
C	1020-024-037	1
D	1020-024-013	1
E	0001-025-000	2
F	0011-224-000	4
H	0004-156-000	2
K	0016-028-000	2
L	1020-024-033	1

1010-091-030

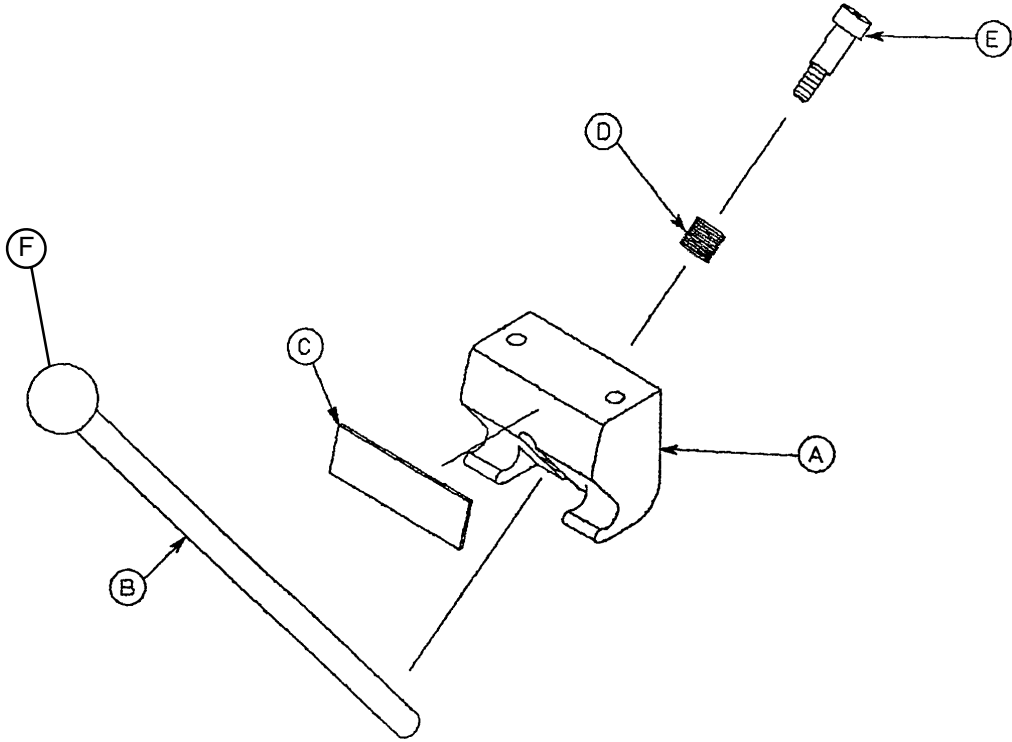


A	1010-024-015	1
B	1010-024-016	1
C	1010-091-014	1
D	0038-263-000	1
E	0008-017-000	1
F	1010-026-046	1

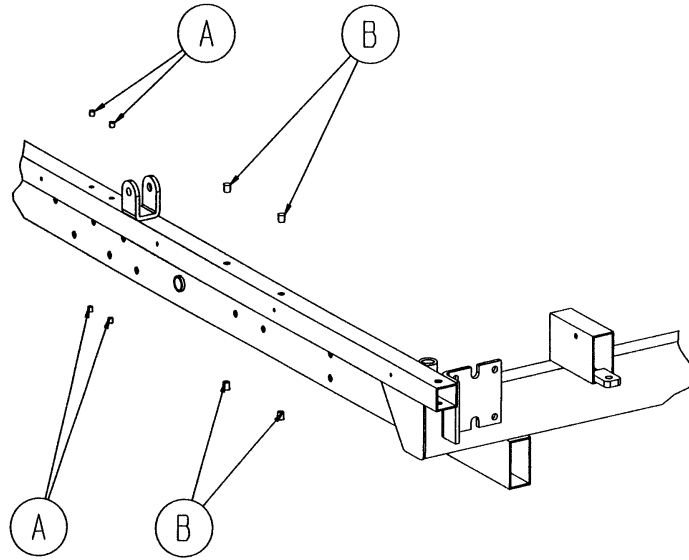


A	0037-074-000	8
B	0721-031-065	8

1020-151-000

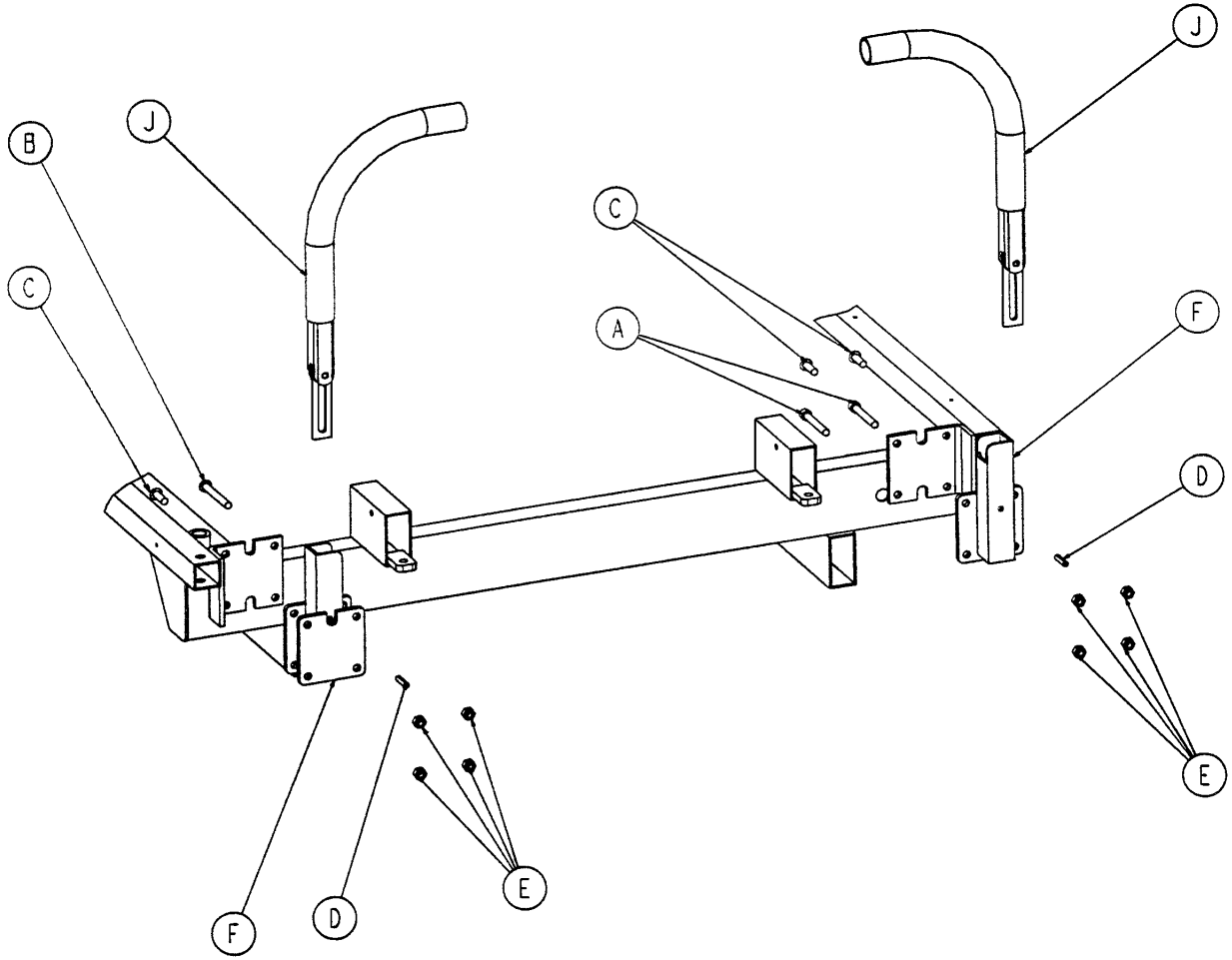


A	0004-009-000	2
B	0004-097-000	3
C	0004-161-000	3
D	0026-011-000	2
E	0016-028-000	8
F	1010-054-070	1
H	1010-054-071	1
J	1211-351-010	2


1211-351-010

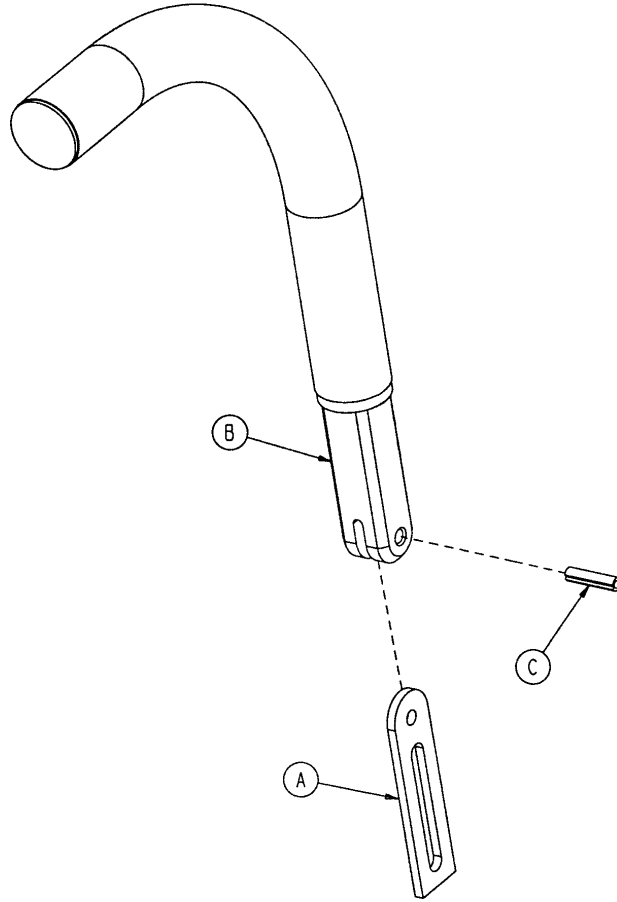

A	0026-118-000	1
B	1010-354-024	1
C	1211-151-018	1

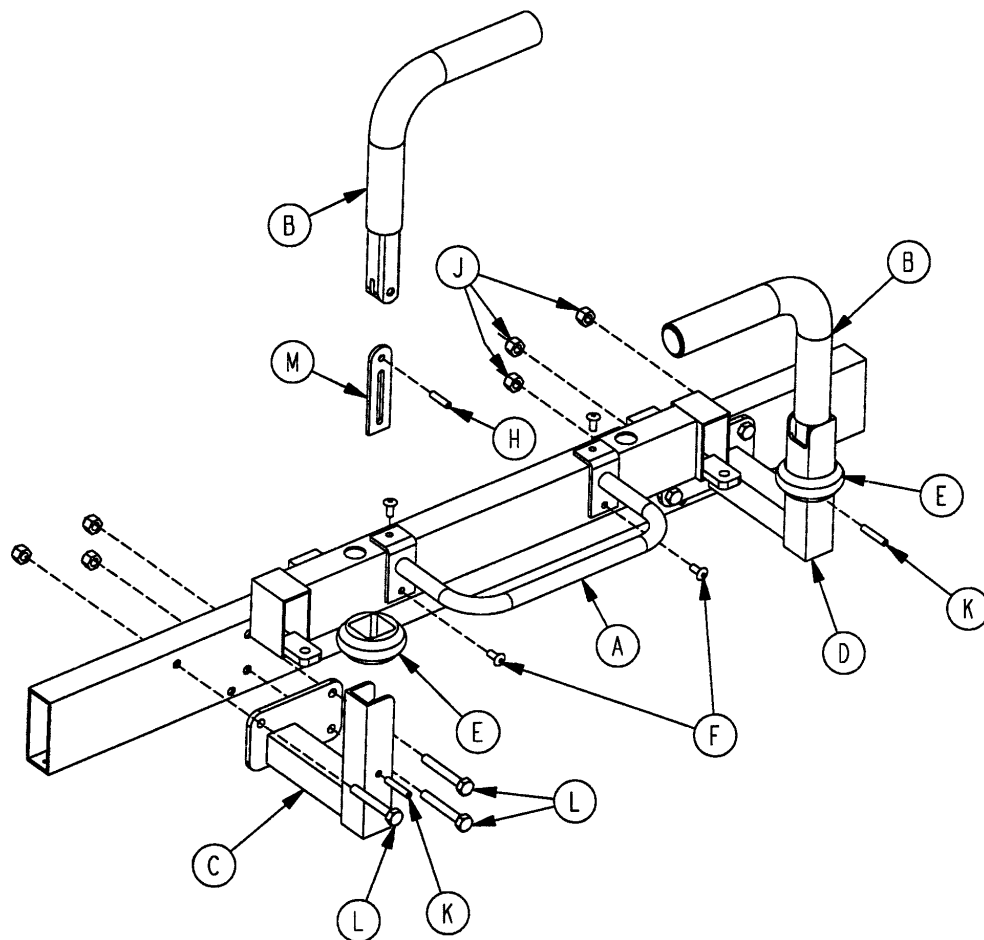



A	1010-051-019	1
B	1000-058-019	2
C	1020-051-015	1
D	1020-051-017	1
E	1010-051-018	2
F	0025-079-000	4
H	0026-010-000	2
J	0016-028-000	6
K	0026-011-000	2
L	0003-078-000	6
M	1010-354-024	2

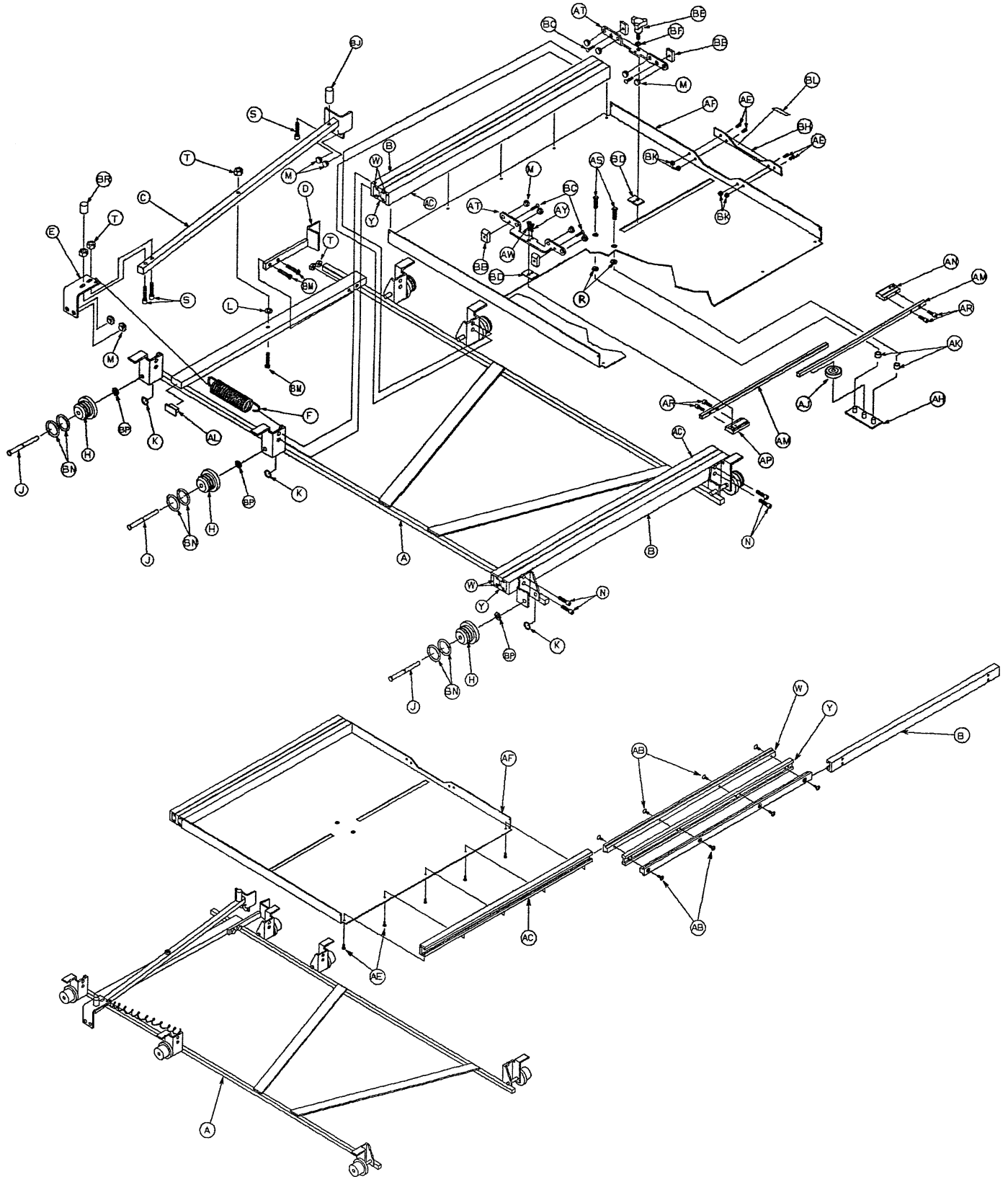

1020-022-010


A	1020-022-044	1	AJ	1020-022-041	1
B	1020-022-011	2	AK	0052-250-000	2
C	1020-022-015	1	AL	1010-032-066	2
D	1020-022-007	1	AM	1020-022-043	2
E	1020-022-036	1	AN	1020-022-058	1
F	0038-253-000	1	AP	1020-022-057	1
H	1020-022-059	6	AR	0004-127-000	4
J	1020-022-025	6	AS	0001-026-000	2
K	0028-098-000	6	AT	1020-022-056	2
L	0011-146-000	1	AW	0004-169-000	1
M	0037-064-000	12	AY	0004-170-000	1
N	0004-136-000	8	BB	1020-022-021	4
P	0004-101-000	18	BC	0001-085-000	4
R	0011-011-000	2	BD	1020-022-055	2
S	0004-026-000	3	BE	0024-051-000	1
T	0016-003-000	5	BF	0011-132-000	1
W	1020-022-013	4	BH	1020-022-029	1
Y	1020-022-012	2	BJ	1010-022-035	1
Z	1020-022-045	2	BK	0016-023-000	4
AB	0001-087-000	16	BL	1020-022-060	1
AC	1020-022-047	2	BM	0004-171-000	3
AD	1020-022-046	2	BN	0045-952-000	12
AE	0004-167-000	14	BP	0014-004-000	6
AF	1020-022-014	1	BR	1010-022-036	1
AH	1020-022-052	1			

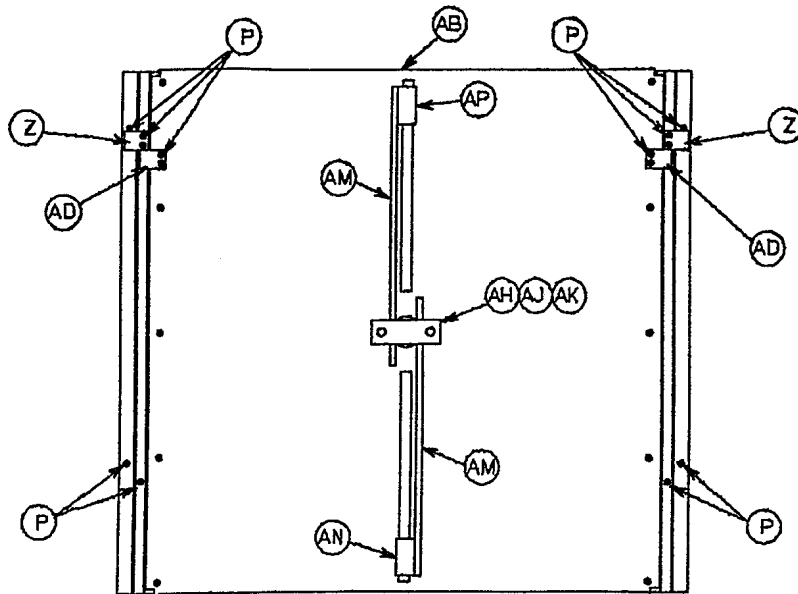

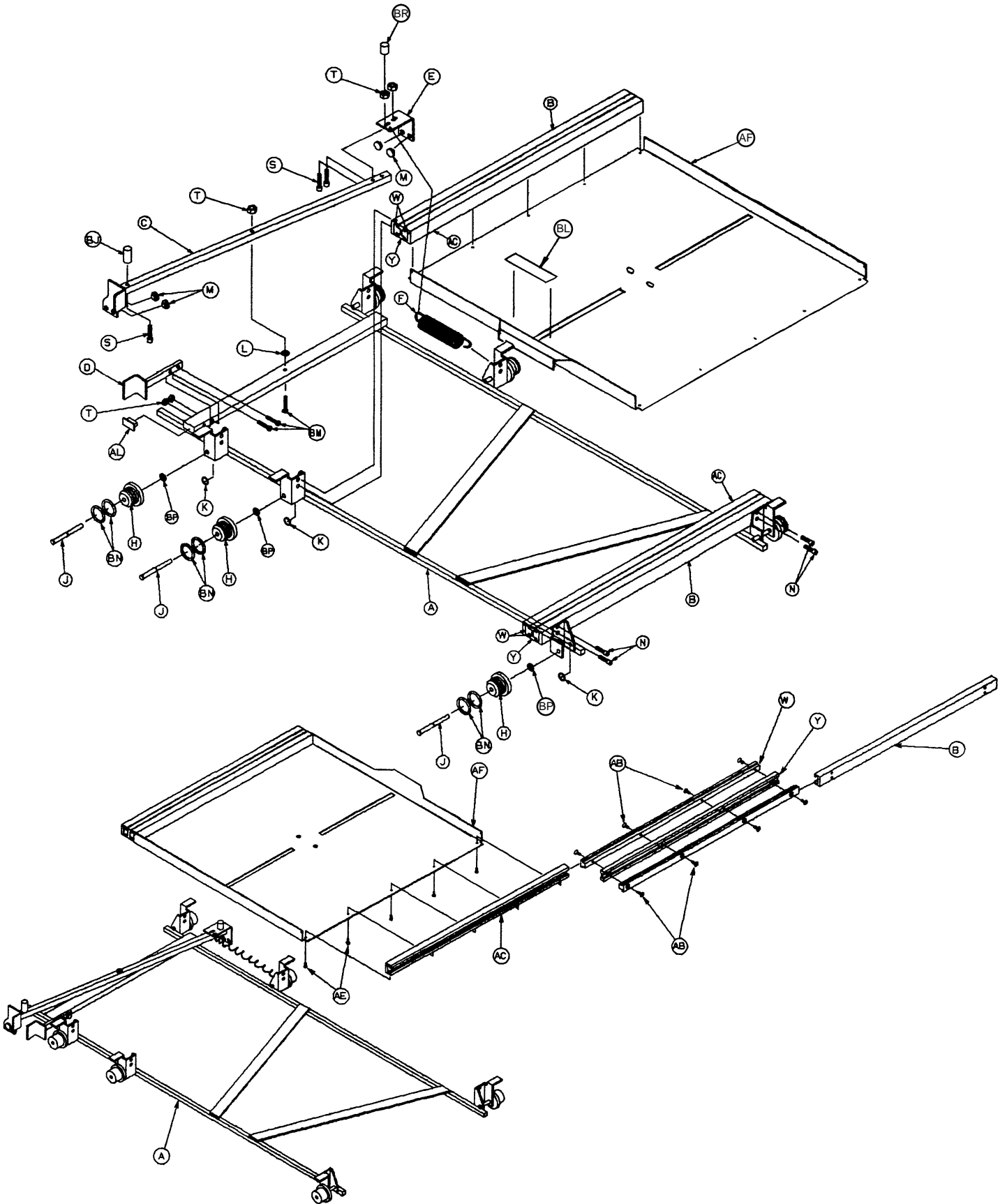


1020-021-010

A	1020-021-044	1	W	1020-022-013	4
B	1020-022-011	2	Y	1020-022-012	2
C	1020-021-015	1	Z	1020-022-045	2
D	1020-021-007	1	AB	0001-087-000	16
E	1020-021-036	1	AC	1020-022-047	2
F	0038-253-000	1	AD	1020-022-046	2
H	1020-022-059	6	AE	0004-167-000	10
J	1020-022-025	6	AF	1020-021-014	1
K	0028-098-000	6	AL	1010-032-066	2
L	0011-146-000	1	BJ	1010-022-035	1
M	0037-064-000	4	BL	1020-022-060	1
N	0004-136-000	8	BM	0004-171-000	3
P	0004-101-000	18	BN	0045-952-000	12
S	0004-026-000	3	BP	0014-004-000	6
T	0016-003-000	5	BR	1010-022-036	1

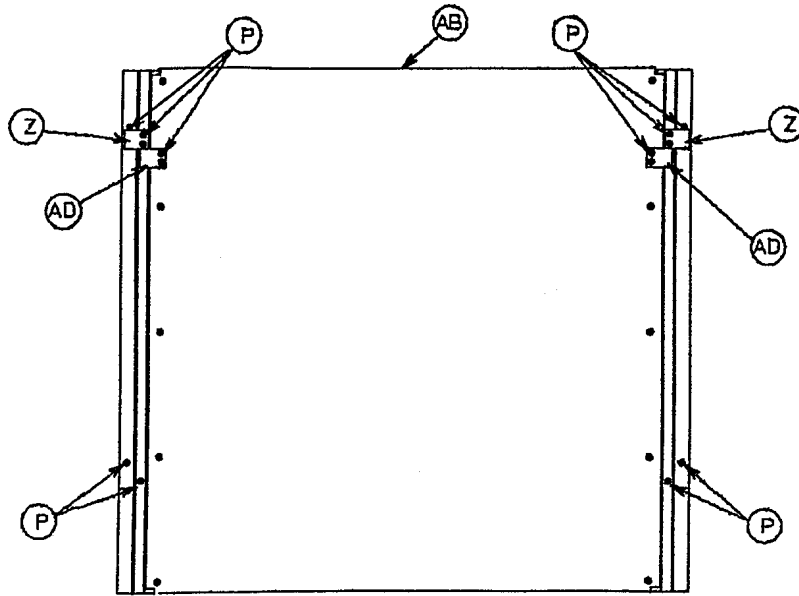


1020-027-000

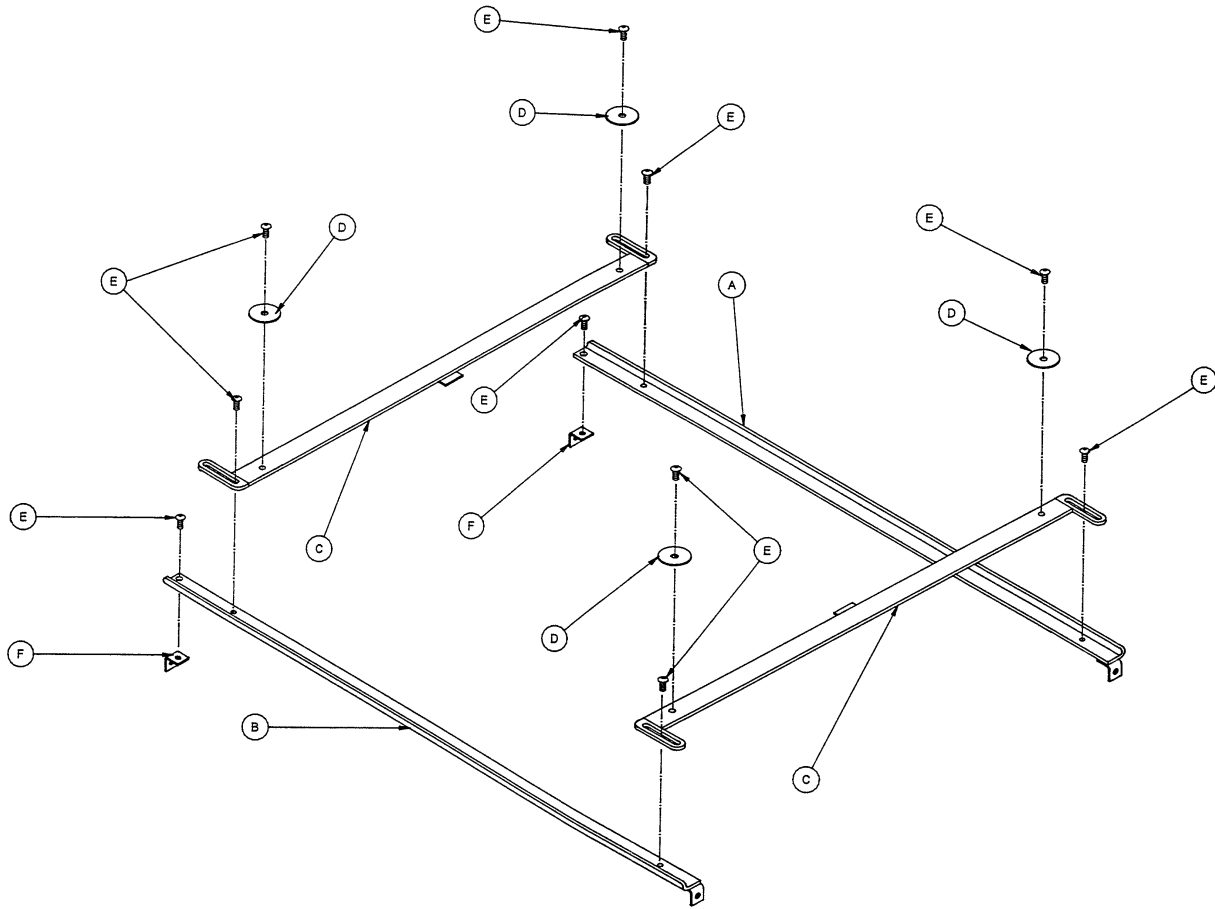


A	1020-027-021	1
B	1020-027-022	1
C	1020-027-011	2
D	0011-271-000	4
E	0004-131-000	10
F	1020-027-024	2

1020-020-000

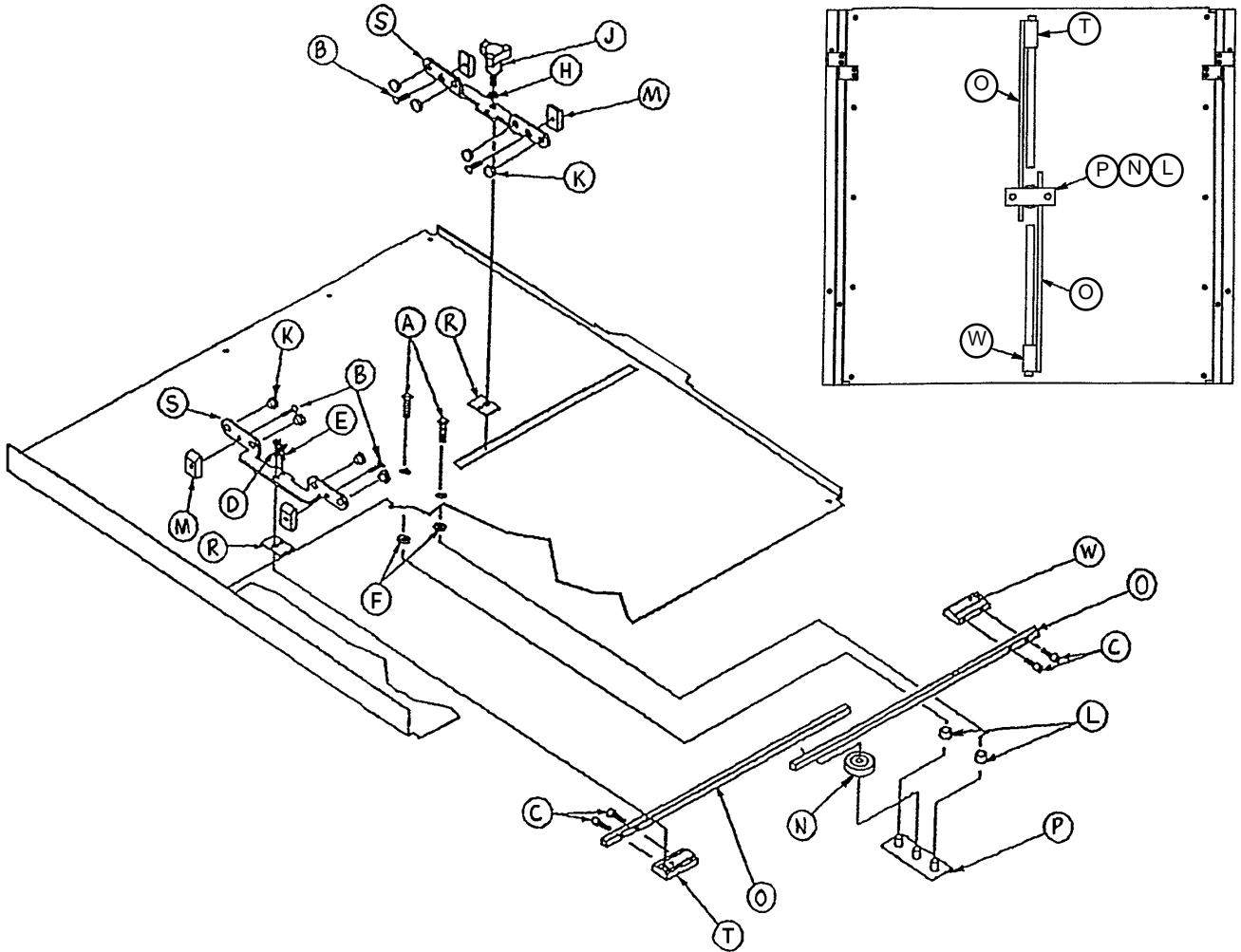


A	0001-022-000	2
B	0001-085-000	4
C	0004-127-000	4
D	0004-169-000	1
E	0004-170-000	1
F	0011-011-000	2
H	0011-132-000	1
J	0024-051-000	1
K	0037-064-000	8
L	0052-250-000	2
M	1020-022-021	4
N	1020-022-041	1
P	1020-022-052	1
R	1020-022-055	2
S	1020-022-056	2
T	1020-022-057	1
W	1020-022-058	1

1020-023-000

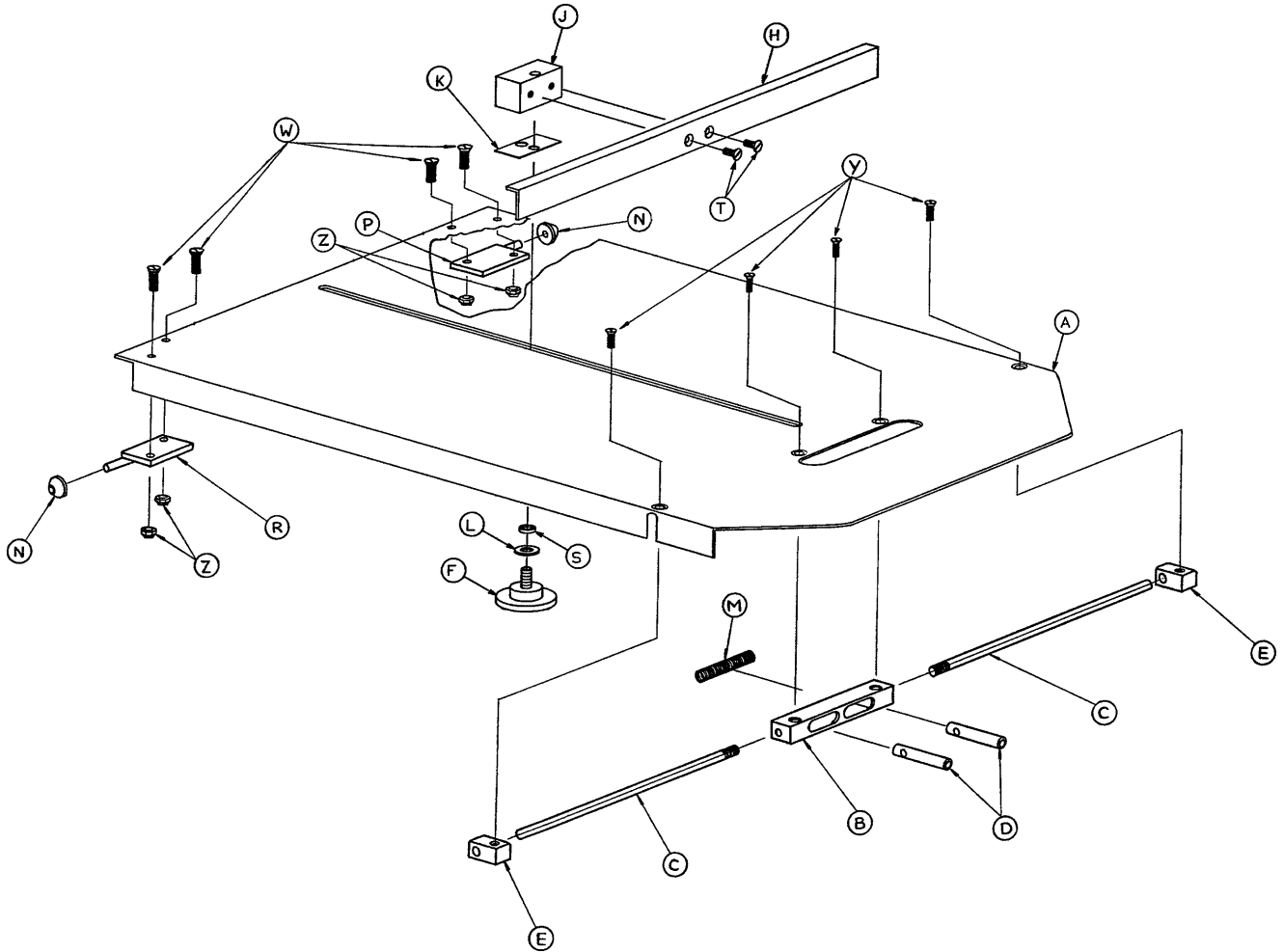


A	1020-023-011	1
B	1020-023-016	1
C	1020-023-025	2
D	0926-023-064	2
E	1010-023-037	2
F	1020-023-021	1
H	1010-023-028	1
J	0926-023-070	1
K	0926-023-069	1
L	0014-003-000	1
M	0038-122-000	1
N	1020-023-026	2
P	1020-023-019	1
R	1020-023-020	1
S	0926-023-071	1
T	0001-020-000	2
W	0004-149-000	4
Y	0001-022-000	4
Z	0016-003-000	4
AA	1010-023-019	1
AB	1020-023-027	1


A	0024-023-000	1
B	0390-003-053	1
C	0393-003-043	1
D	0004-496-000	1

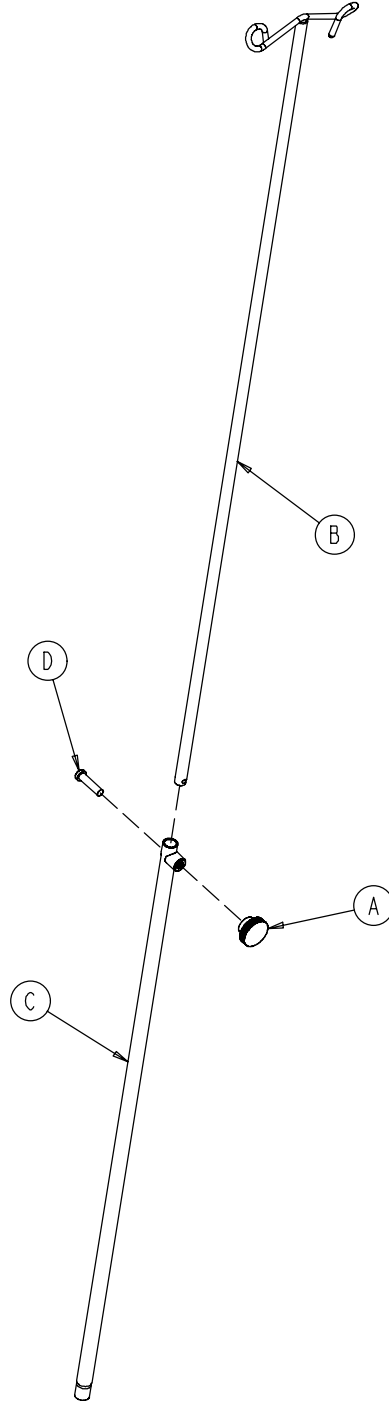

1010-022-100

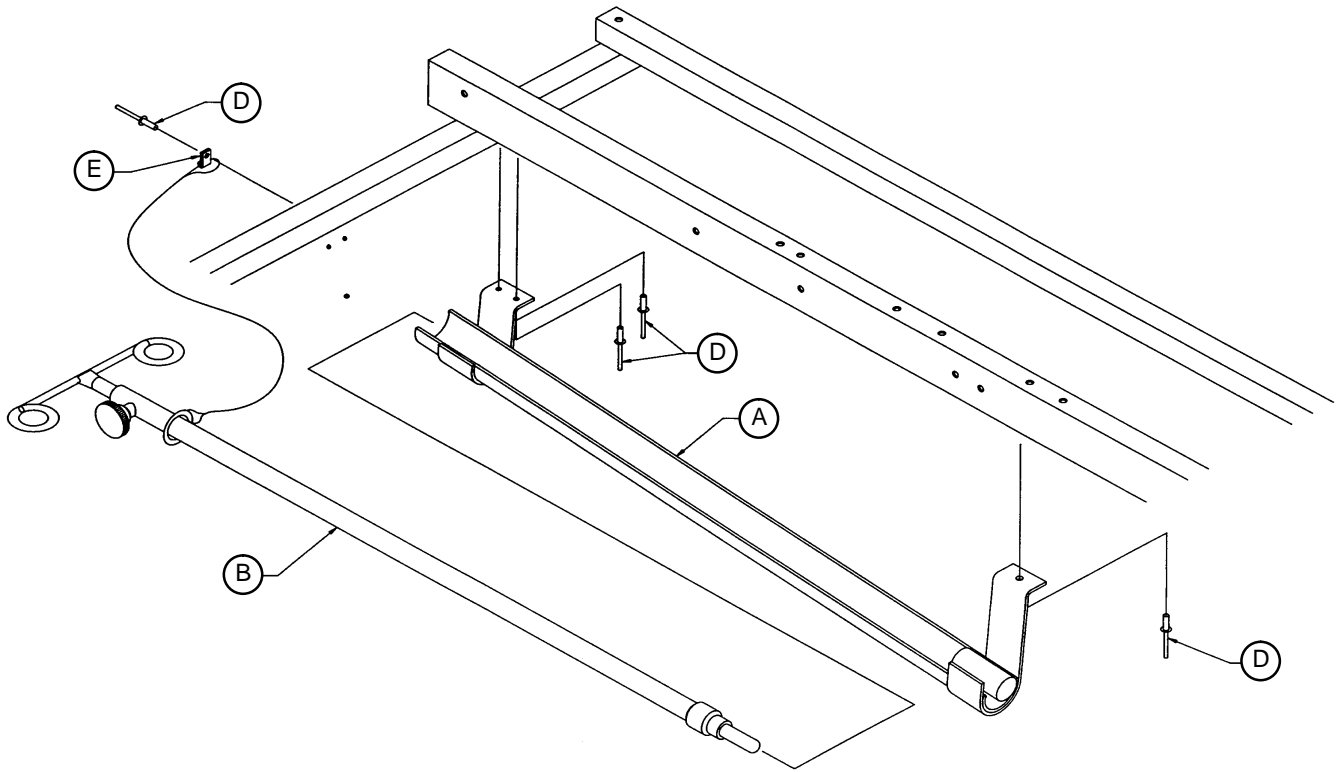


A	1010-022-025	1
B	1001-087-018	1
D	0025-092-000	4
E	0034-022-000	1

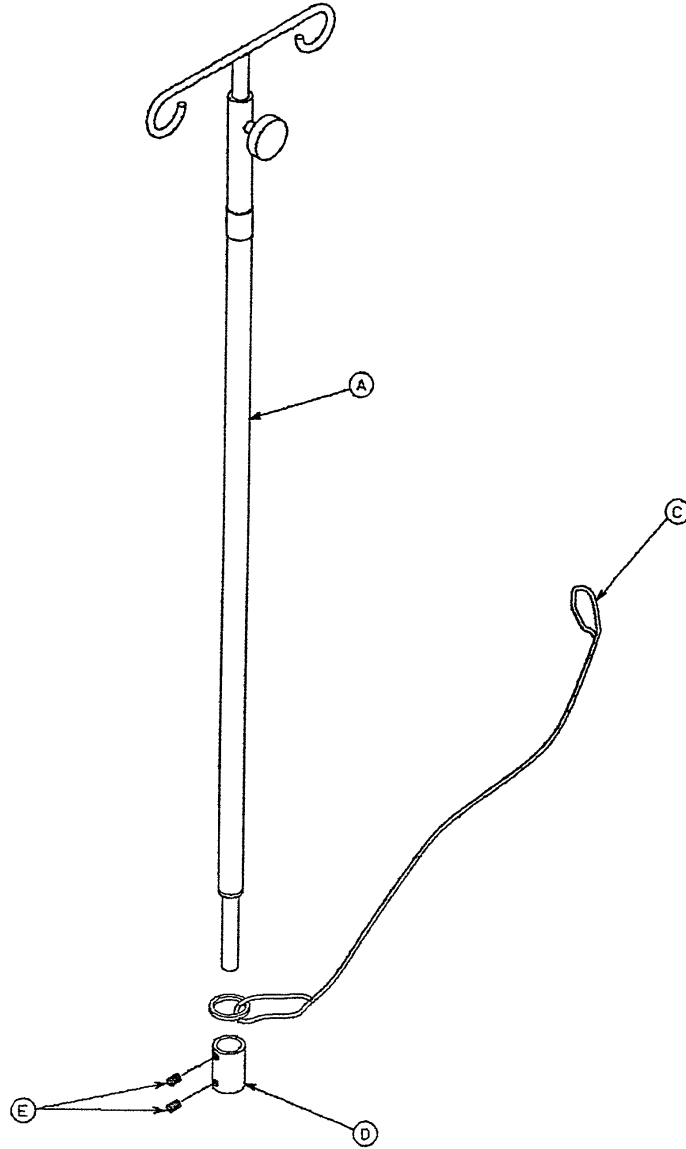


A	0721-047-006	1
C	1010-022-027	1
D	0741-047-014	1
E	0021-016-000	2

1010-080-000 & 1010-081-000

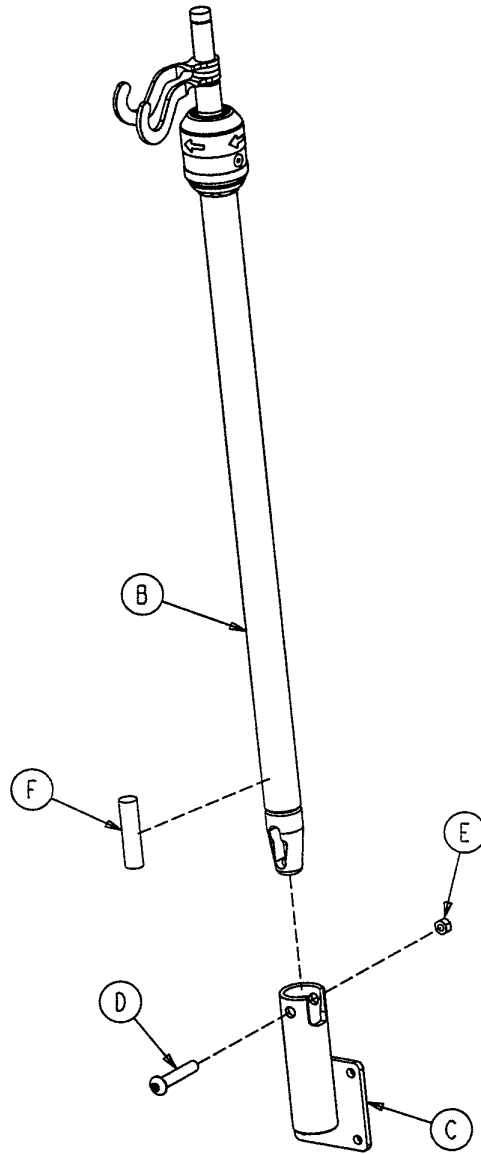


B	1211-210-010	1
C	1001-259-111	1
D	0004-199-000	1
E	0016-011-000	1
F	1001-059-042	1

1211-210-010

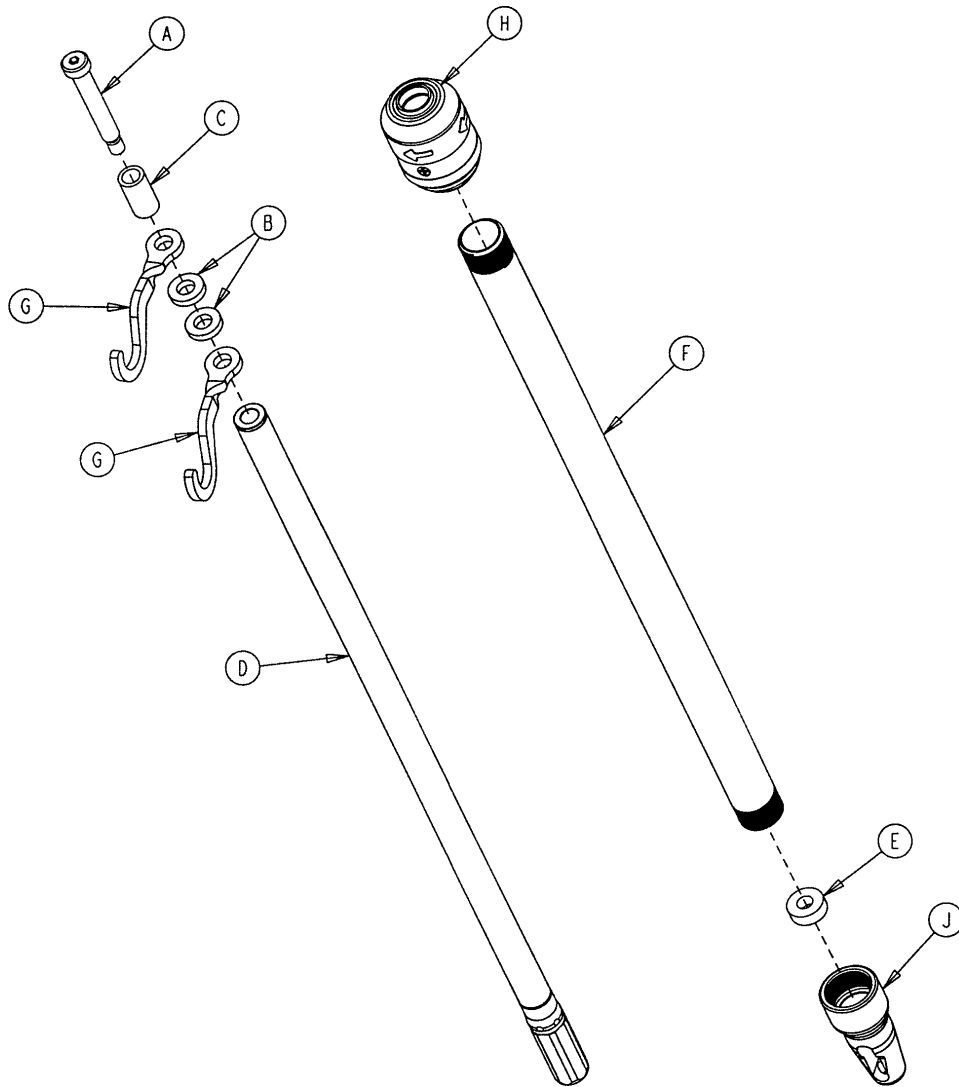


A	0008-031-000	1
B	0052-017-000	2
C	0926-400-162	1
D	1211-210-029	1
E	1001-359-013	1
F	1001-159-028	1
G	1010-259-016	2
H	1211-110-026	1
J	1001-359-112	1

1211-210-026

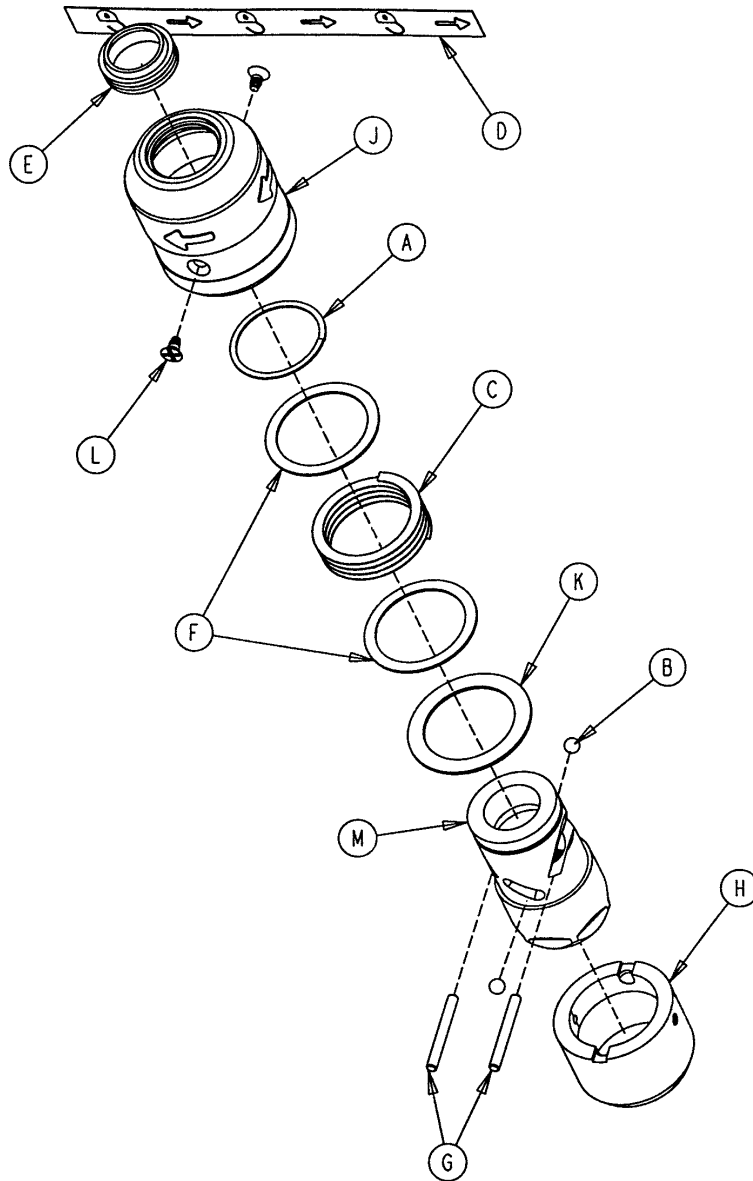


A	0028-167-000	1
B	0031-004-000	2
C	0038-392-000	1
D	1211-091-034	1
E	1211-110-018	1
F	1211-110-020	2
G	1211-110-021	2
H	1211-110-022	1
J	1211-110-024	1
K	1211-110-035	1
L	1211-110-036	2
M	1211-210-023	1

1020-062-000 & 1020-061-000

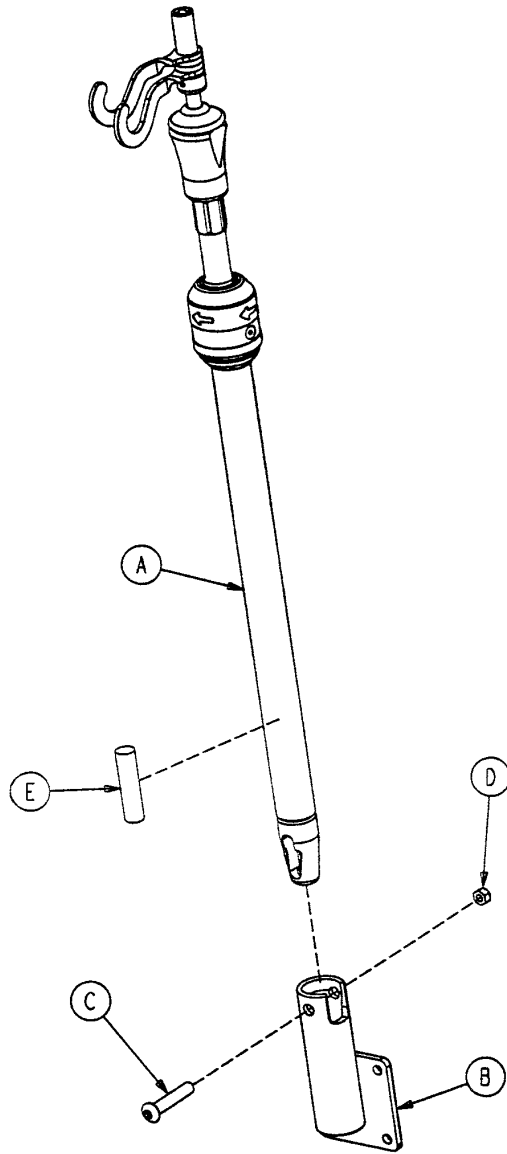


A	1211-211-010	1
B	1001-259-111	1
C	0004-199-000	1
D	0016-011-000	1
E	1001-059-042	1

1211-211-010

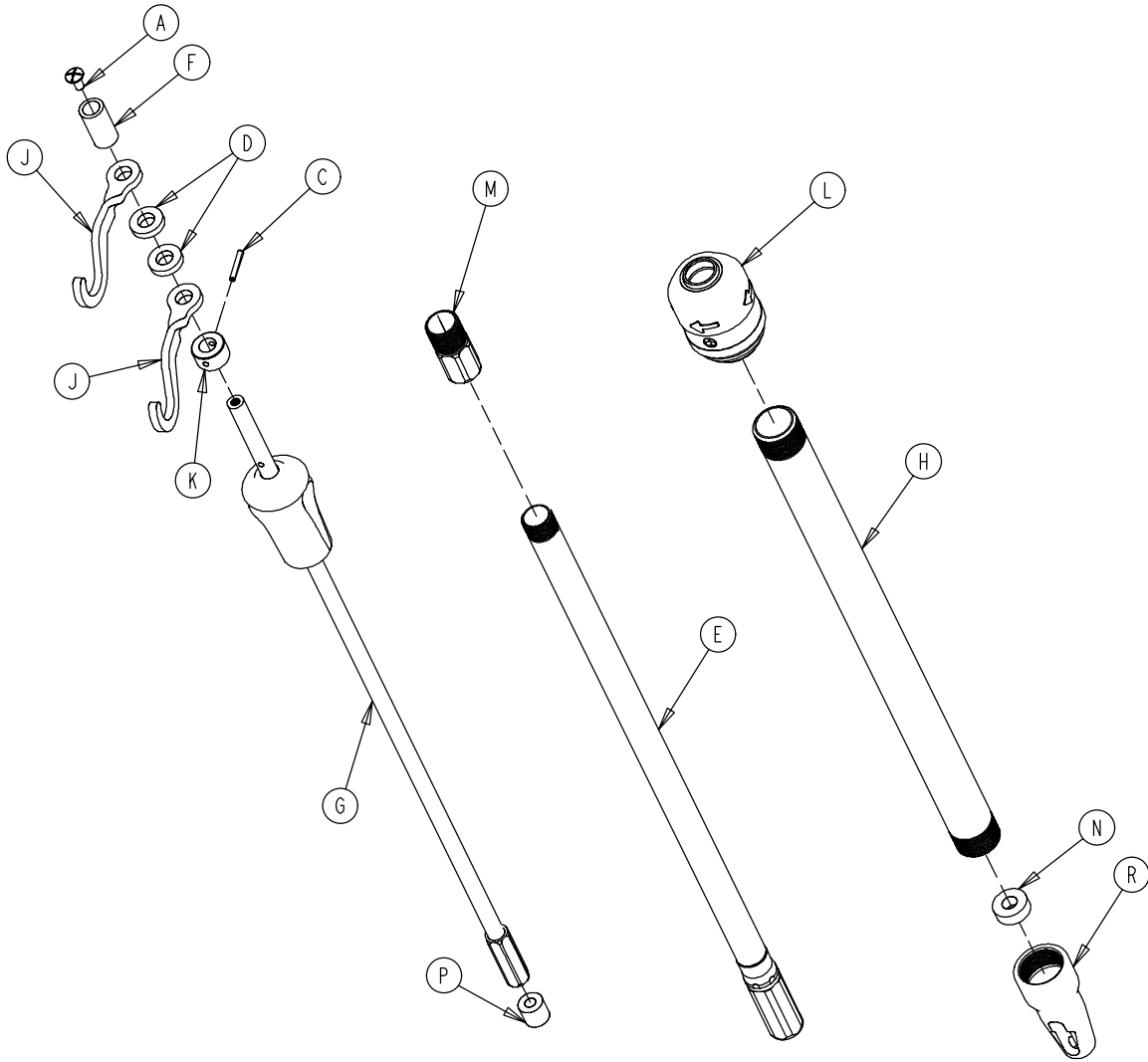


A	0007-004-000	1
C	0026-076-000	1
D	0052-017-000	2
E	1211-210-031	1
F	0926-400-162	1
G	1211-110-032	1
H	1001-161-023	1
J	1010-259-016	2
K	1010-061-014	1
L	1211-210-026	1
M	1211-110-016	1
N	1001-359-013	1
P	1001-259-014	1
R	1001-359-112	1

1211-110-032

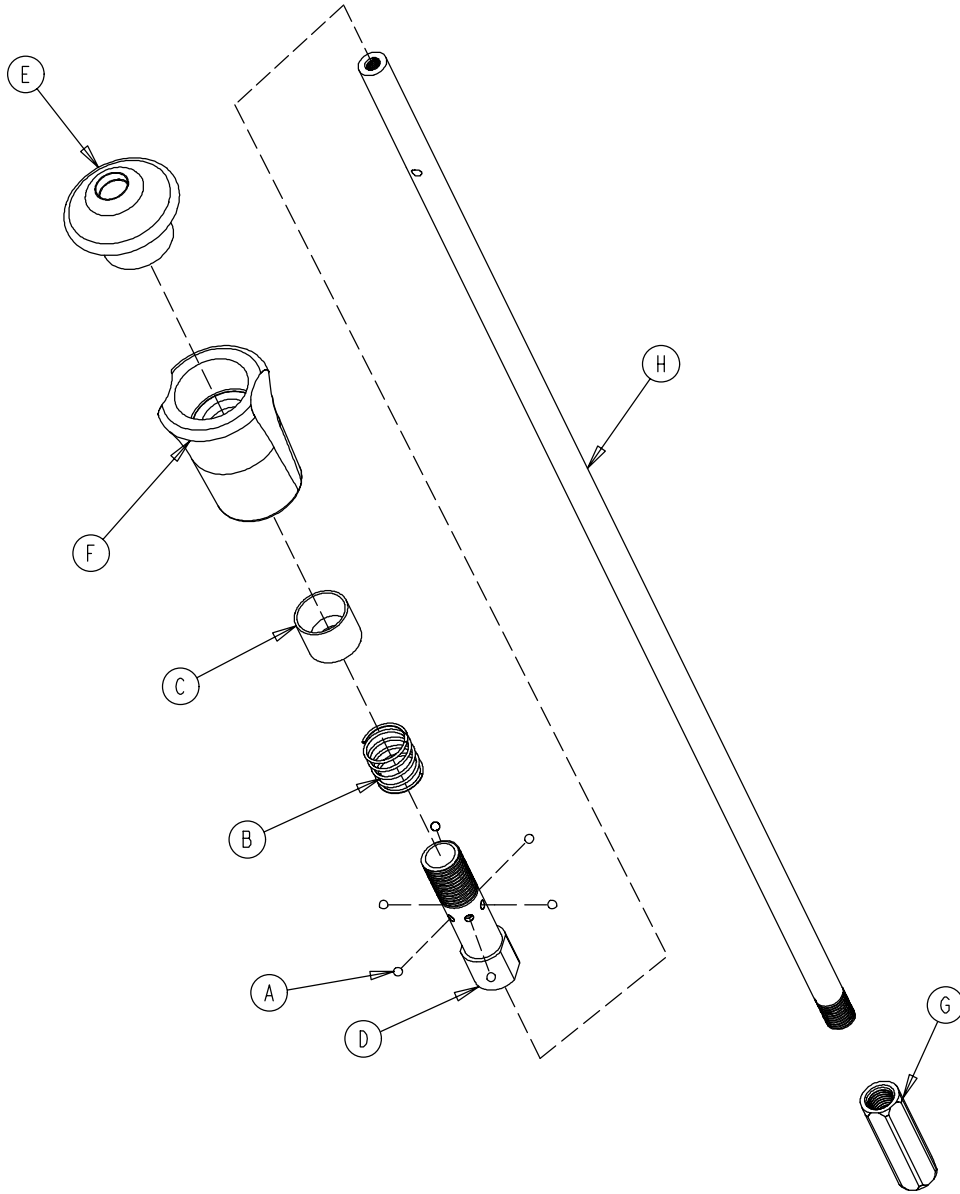


A	0031-021-000	6
B	0038-303-000	1
C	1010-061-013	1
D	1010-061-016	1
E	1010-061-017	1
F	1010-061-018	1
G	1211-110-117	1
H	1211-110-033	1

0741-034-100

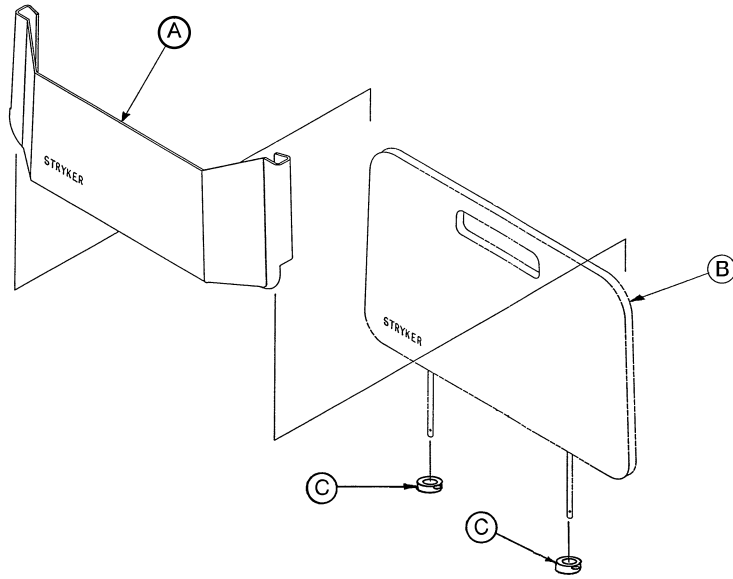


A	0946-028-011	1
B	0741-029-018	1
C	0042-006-000	2

1020-030-000

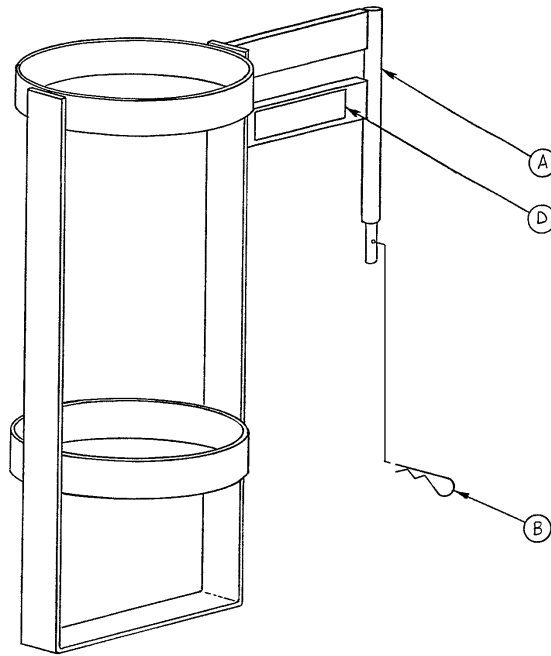


A	1020-030-011	1
B	0027-012-000	1
D	1020-030-017	1

0741-048-000

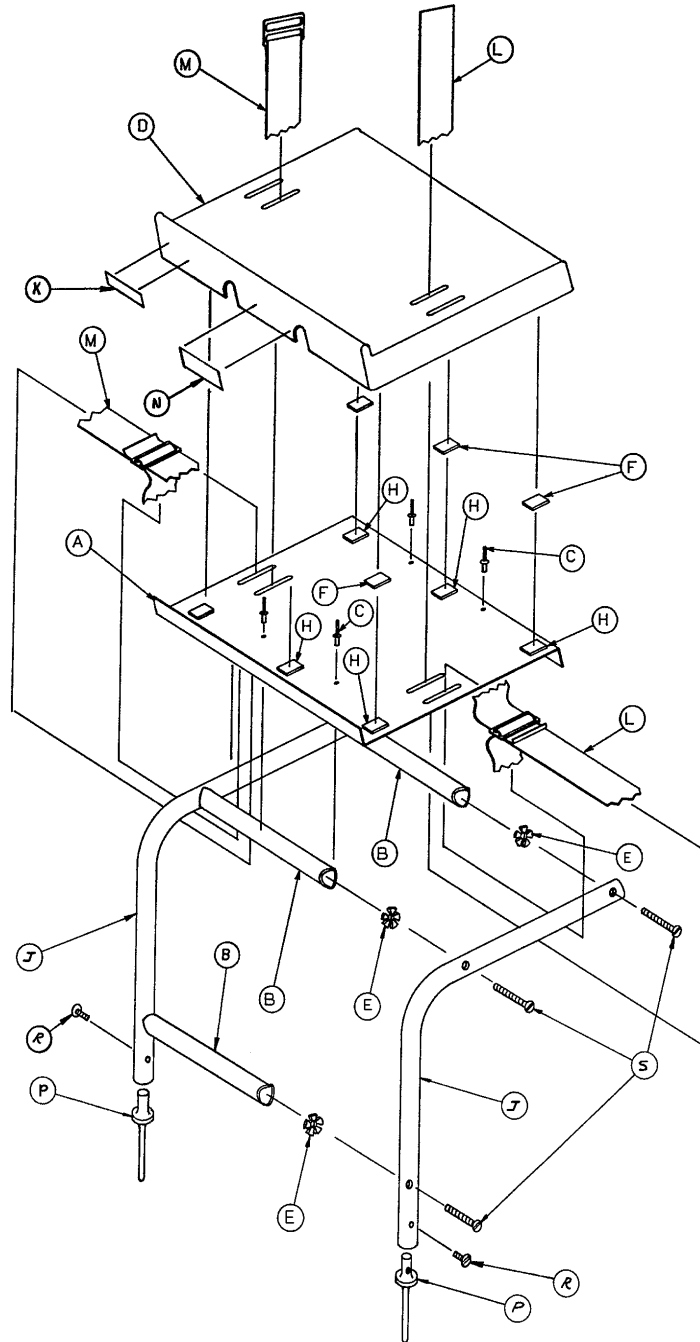


A	0946-039-004	1	K	0741-048-012	1
B	0926-039-016	3	L	0390-057-012	1
C	0025-055-000	4	M	0390-057-013	1
D	0926-039-009	1	N	0946-001-283	1
E	0018-006-000	6	P	0741-048-020	2
F	0029-008-000	6	R	0002-044-000	2
H	0029-010-000	6	S	0002-105-000	6
J	0946-039-003	2			

1010-050-100

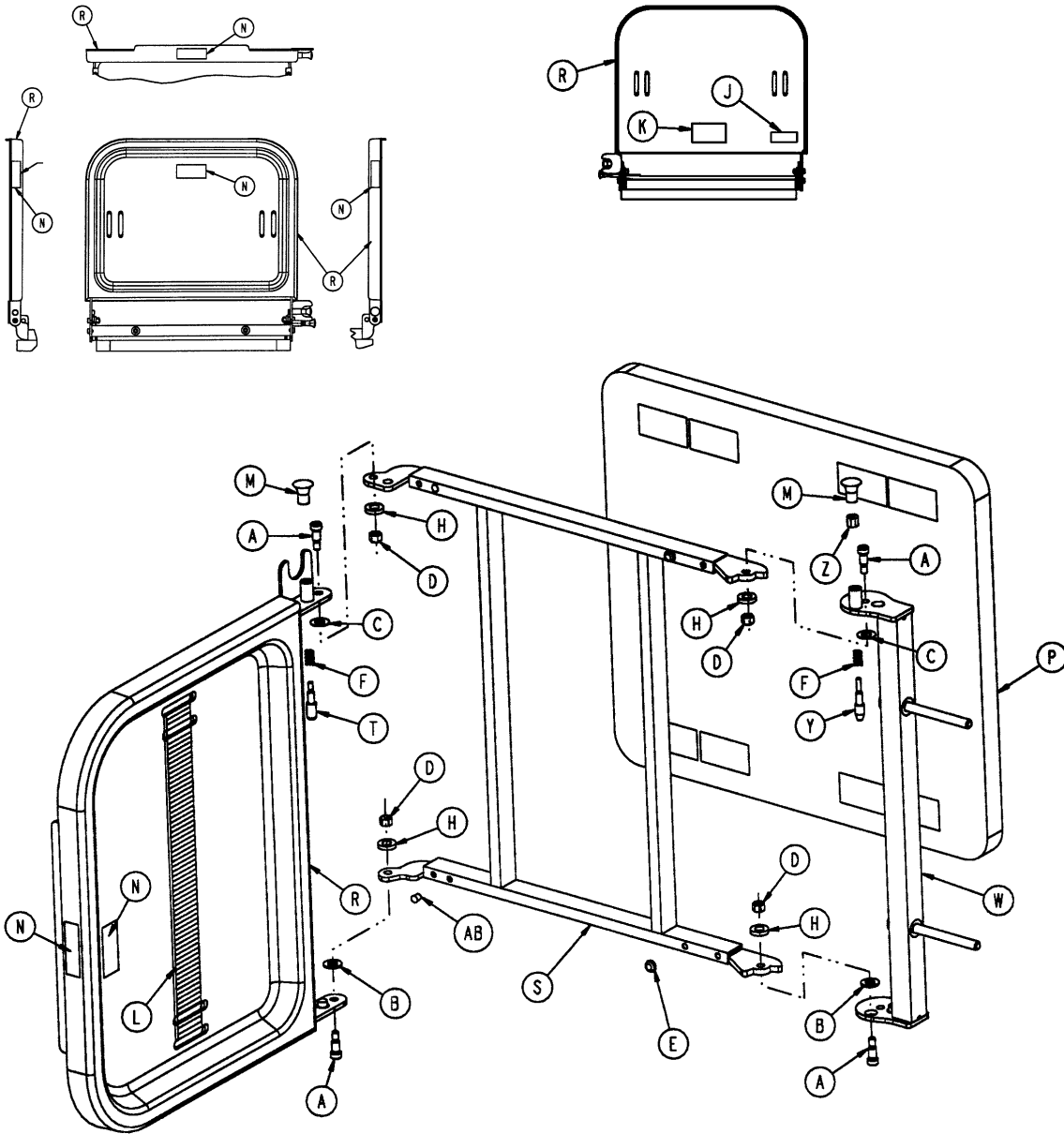


A	0008-049-000	4	M	1010-050-050	2
B	0014-020-000	2	N	1010-091-057	4
C	0014-021-000	2	P	1010-050-205	1
D	0016-028-000	4	R	1010-050-218	1
E	0037-052-000	2	S	1010-050-225	1
F	0038-133-000	2	T	1010-050-242	1
H	0052-017-000	4	W	1010-050-246	1
J	1010-050-017	1	Y	1010-050-248	1
K	1010-091-020	1	Z	1010-050-250	1
L	1010-050-021	1	AB	0721-031-065	2

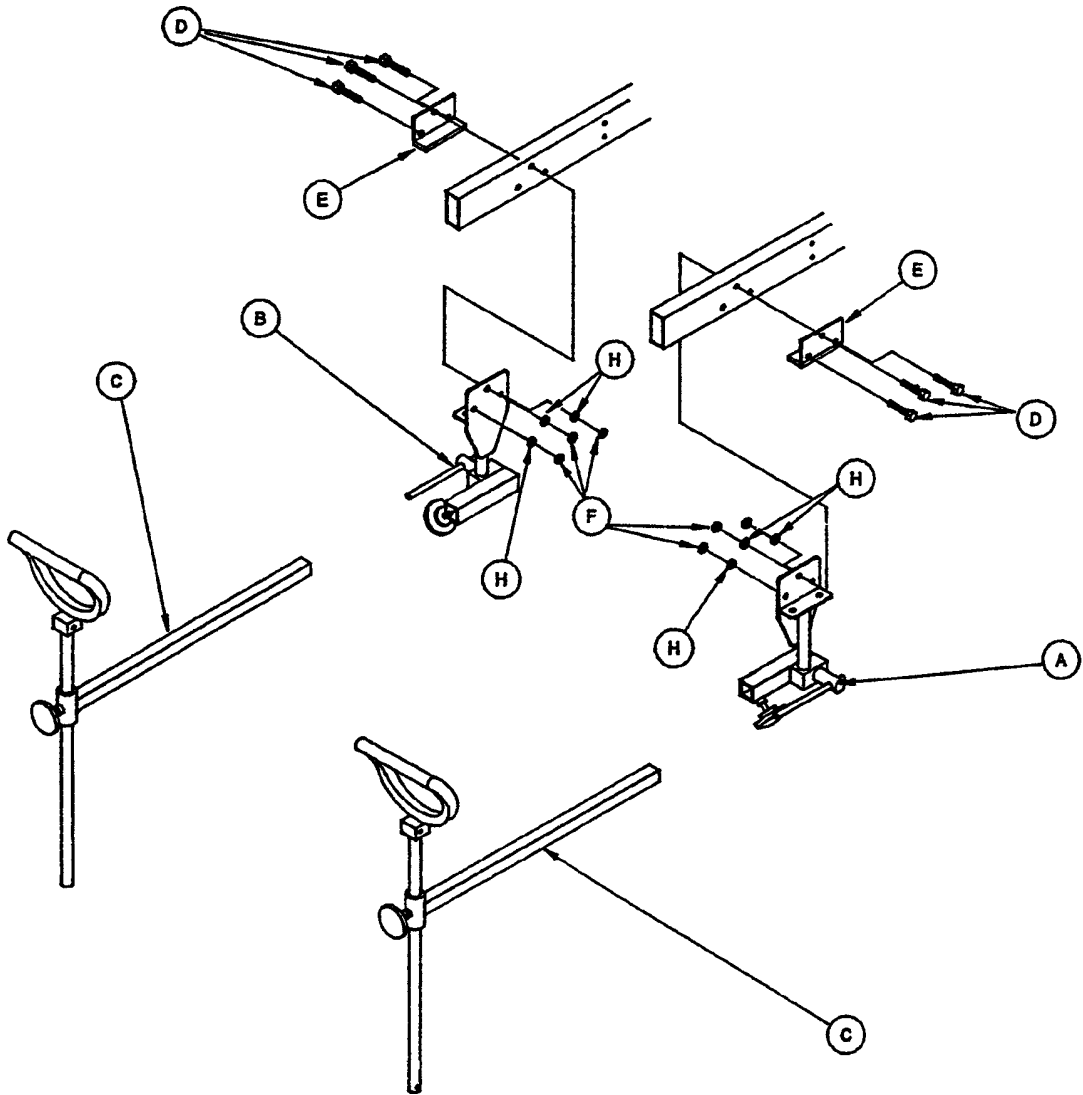


A	1010-055-015	1
B	1010-055-025	1
C	1010-055-029	2
D	0003-078-000	6
E	1020-055-015	2
F	0016-028-000	6
F	0011-158-000	6

1010-055-015 & 1010-055-025

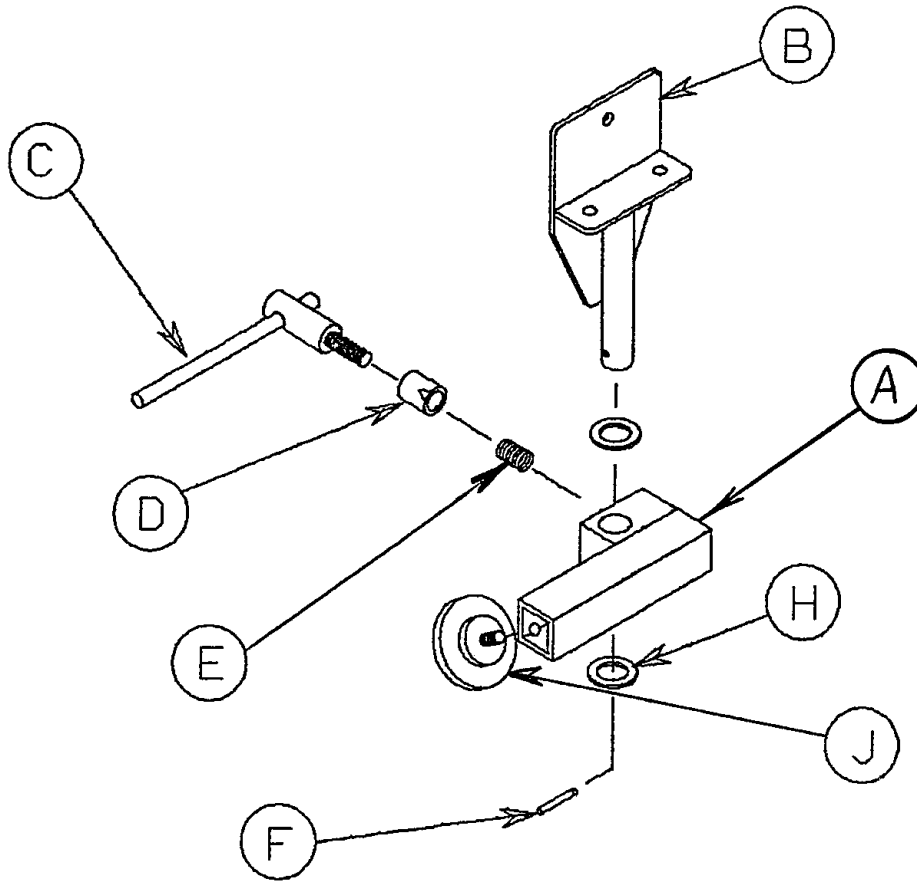


A	1010-055-016	1
	1010-055-026	1
B	1010-055-020	1
C	0650-002-032	1
D	0966-051-019	1
E	0038-171-000	1
F	0026-007-000	1
H	0014-040-000	2
J	0024-011-000	1

1010-055-029

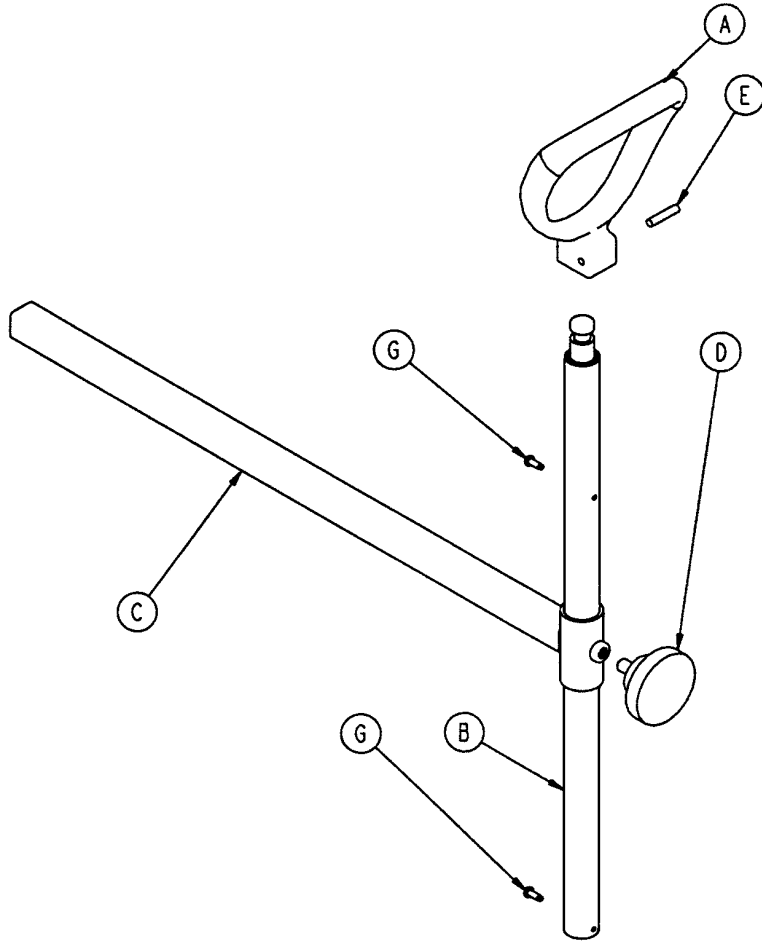


A	0390-020-013	1
B	0390-020-009	1
C	1010-055-030	1
D	0024-011-000	1
E	0026-043-000	1
G	0022-008-000	2

1020-070-000

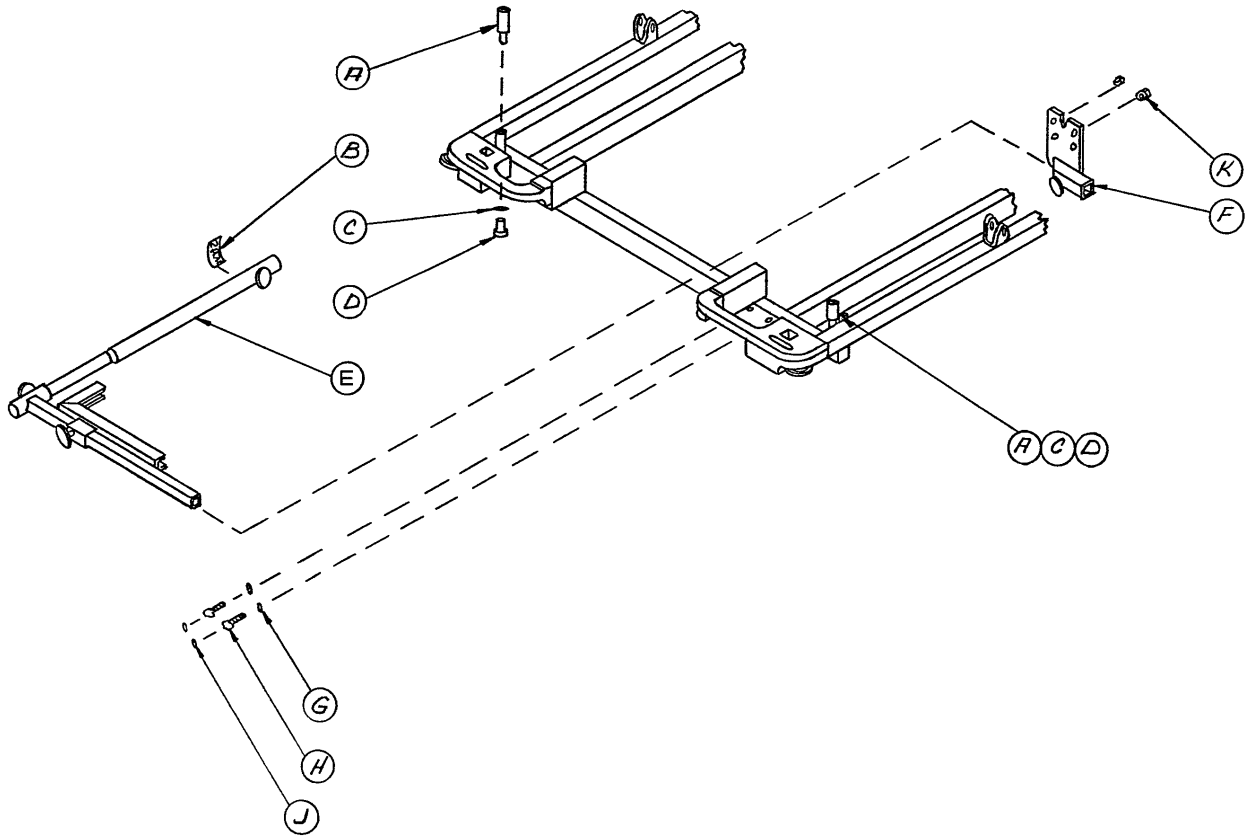




A	1010-070-025	2
B	1020-070-018	1
C	0011-003-000	2
D	0004-091-000	2
E	1010-070-010	1
F	1020-070-019	1
G	0037-056-000	2
H	0003-047-000	2
J	0037-055-000	2
K	0016-028-000	2

1010-070-010

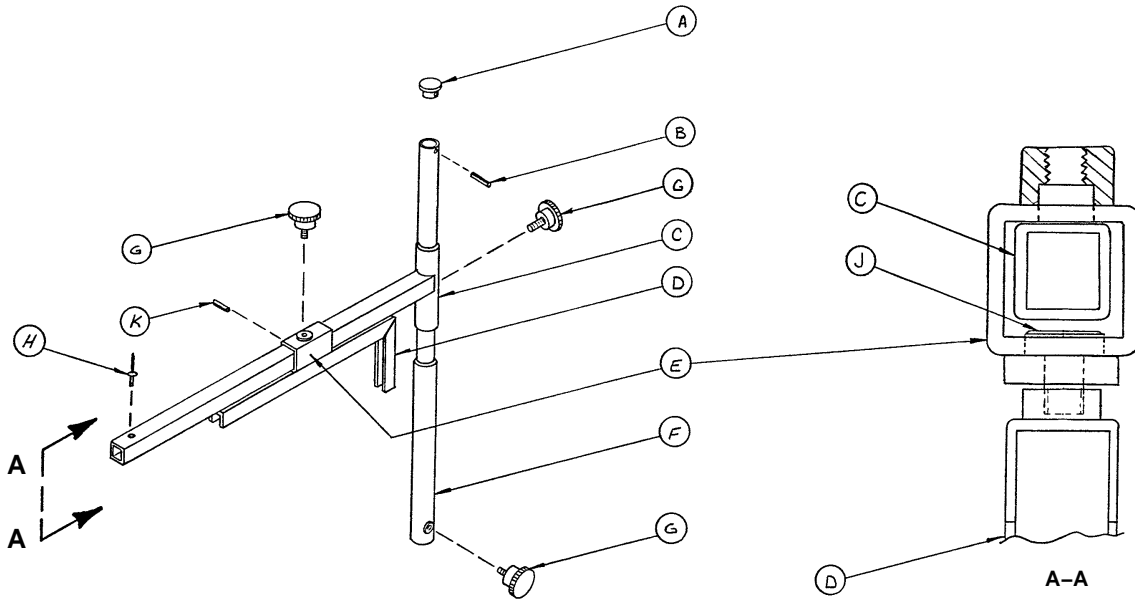

A	1010-070-033	1
B	0026-006-000	1
C	1010-070-034	1
D	1010-070-042	1
E	1010-070-050	1
F	1010-070-030	1
G	1010-070-045	3
H	0025-055-000	1
J	1010-070-044	1
K	0026-005-000	1

1020-70-19

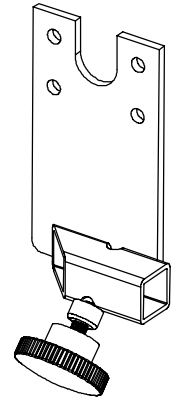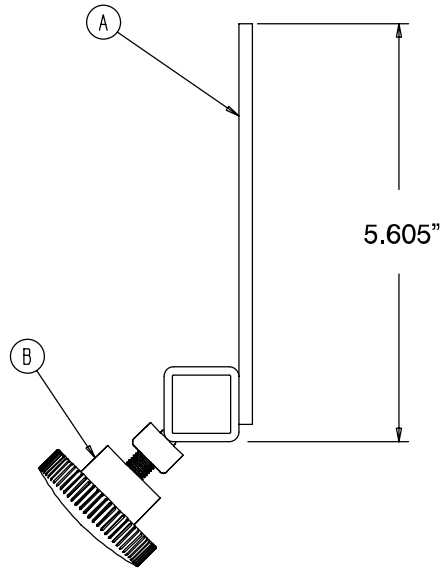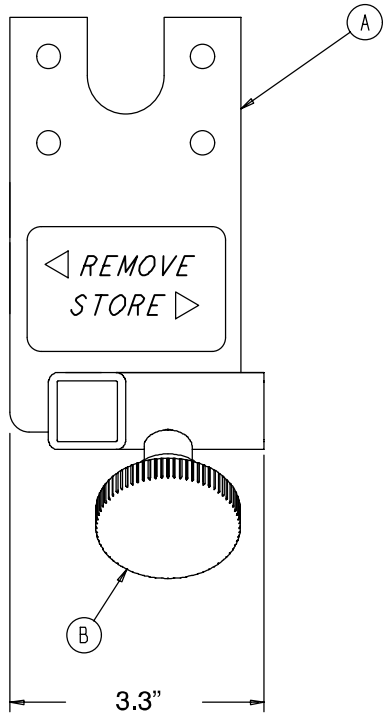

A	1020-70-20	1
B	1010-70-45	1
C	1010-70-23	1

Addresses / Adressen / Anschriften / Adresse / Indirizzi

Direcciones / Endereços / Adresser

Stryker Howmedica/Osteonics Belgium
Ikaros Business Park – Fase II
Ikaroslaan 12
1930 Zaventem
Brussels
Tel. +3227179210
Fax +3227179249

Stryker Osteonics Lugano
Via Della Posta
P.O. Box 254
6934 Bioggio
Tel. +41916104410
Fax +41916104411

Stryker Howmedica Osteonics– Netherlands
Marinus van Meelweg 17 / Postbus 8747
5657 EN
Eindhoven
Nederland
Tel. +31402922522
Fax +31402922566

Stryker UK Limited
Stryker House
Hambridge Road
Newbury, Berkshire
RG14 5EG
England
Tel. +441635556500
Tel. +441635262400
Fax +441635556551

Stryker France SA
13, Rue de la Perdrix
Z.A.C. Paris Nord II
93290 Tremblay–En–France
France
B.P.50040
95946 Roissy Charles–de–Gaulle
France
Tel. +33148175000
Fax +33148632175

Stryker Howmedica Iberica S.L.
Manuel Tovar, 35
28034 Madrid
España
Tel. +34917283500
Tel. +34913582044
Fax +34913580748
VAT no ESB – 82173451

Stryker Howmedica Osteonics GmbH
Gewerbeallee 18
45478 Mülheim a.d. Ruhr
Deutschland
Tel. +49208999060
Fax +492089990666

Stryker Denmark
filial of Stryker AB
Postbox 772
1532 Copenhagen
Denmark
Tel. +4533936099
Fax +4533932069

Stryker Polska Sp. zo.o.
ul. Kolejowa 19/21
01–217 Warszawa
Poland
Tel. +48226312013
Tel. +48226312014
Fax +48226312015

Stryker MedSurg Group
Middle East/Africa Division
Suite 202, Al Masood Tower
Hamdan St.
P.O. Box 26589
Abu Dhabi, United Arab Emirates
Tel. +97126312145
Fax +97126313698

Stryker Osteonics (Pty) Ltd
3 Susan Street
Strydom Park
Johannesburg, 2194–South Africa
Tel. +27117914644
Fax (main) +27117914696
Fax (orders) +27117914697

Stryker Trauma S.A
5, Chemin des Aulx
1228 Plan–les–Ouates
Case Postale 725
1212 Grand–lancy 1
Tel. +41228840111
Fax +41228840199



European Representative

Stryker EMEA RA/QA Director
Stryker France
ZAC Satolas Green Pusignan
Av. De Satolas Green
69881 MEYZIEU Cedex
France

stryker[®]
Medical

www.strykermedical.com



DH 9/03 1020-600-271 REV A