

IMPORTANT
File in your
maintenance
records

stryker[®]
Medical

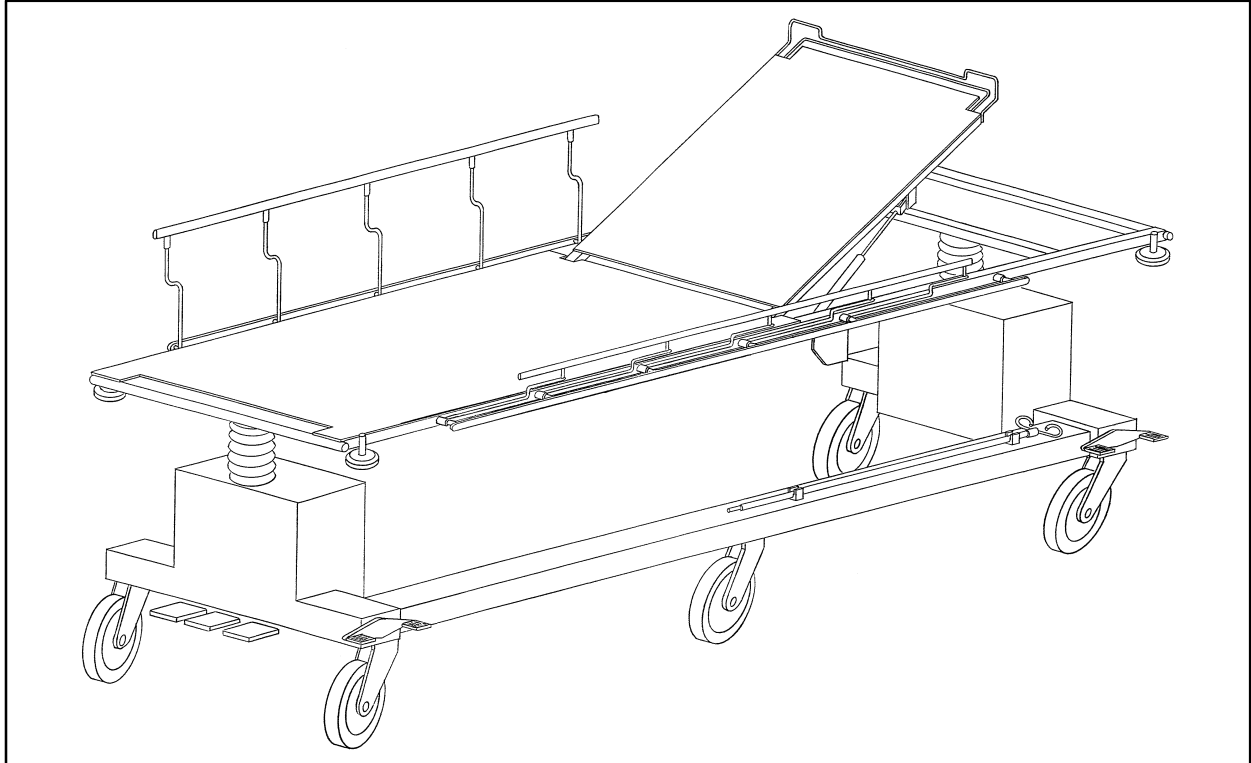

1075 Fluoroscopy Stretcher

OPERATIONS AND MAINTENANCE MANUAL

For Stretchers with serial numbers from 93063494 through 9505031119

For Parts or Technical Assistance:
1-800-327-0770

Table of Contents

- INTRODUCTION 2
- SPECIFICATIONS 2
- WARNING / CAUTION / NOTE DEFINITION 2
- BASE PEDAL OPERATION 3
- SIDERAIL OPERATION 4
- FOWLER OPERATION 4
- I.V. POLE OPERATION 4
- PREVENTATIVE MAINTENANCE 5
- FIELD REPLACEMENT PARTS 6
- LIMITED WARRANTY
 - Obtaining Parts and Service 7
 - Supplemental Warranty Coverage 7
 - Return Authorization 8
 - Freight Damage Claims 8

Introduction

INTRODUCTION

This manual is designed to assist you with the operation and maintenance of the 1075 Fluoroscopy Stretcher. Read it thoroughly before using the stretcher or beginning any maintenance on it.

SPECIFICATIONS

Maximum Weight Capacity	500 pounds
Overall Stretcher Length/Width	88" x 30.5"
Patient Surface Length/Width	84" x 28.5"
Minimum/Maximum Stretcher Height	27"/36"
Fowler Angle	0° to 90°
Trendelenberg/Reverse Trendelenberg	-12° to +12°
Fluoroscopy Window	18.8" x 76"
C-Arm Access Width	66"

WARNING / CAUTION / NOTE DEFINITION

The words WARNING, CAUTION and NOTE carry special meanings and should be carefully reviewed.

WARNING

The personal safety of the patient or user may be involved. Disregarding this information could result in injury to the patient or user.


CAUTION

These instructions point out special procedures or precautions that must be followed to avoid damaging the equipment.


NOTE

This provides special information to make maintenance easier or important instructions clearer.


Operation Guide

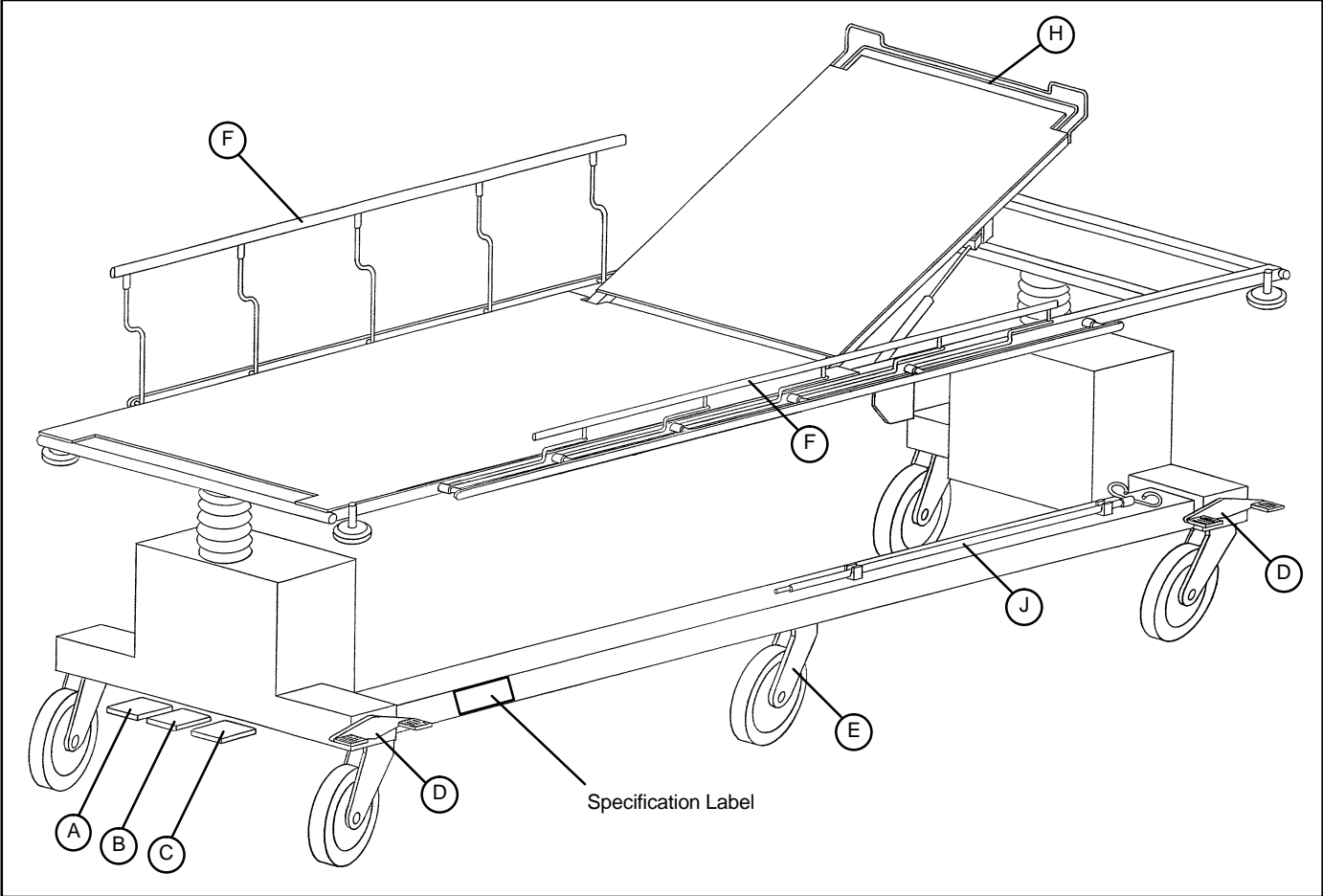

BASE PEDAL OPERATION

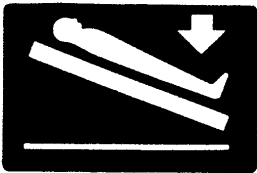


(A) Press to lower foot end (Reverse Trendelenberg).

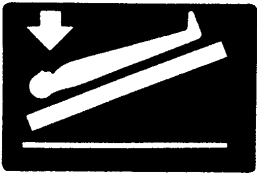

(B) Press to lower head end (Trendelenberg).

Lower both ends of the stretcher by pressing down on both Trendelenberg pedals at the same time.

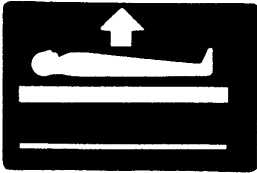


(C) Pump pedal repeatedly to raise litter.

Operation Guide

BASE PEDAL OPERATION (CONTINUED)

The Brake/Steer pedals (D) are located on the base at both ends of the bed on the patient's left. Push the left side of the pedal fully down to set the stretcher brakes. Push the right side of the pedal fully down to activate the fifth wheel steering (E).

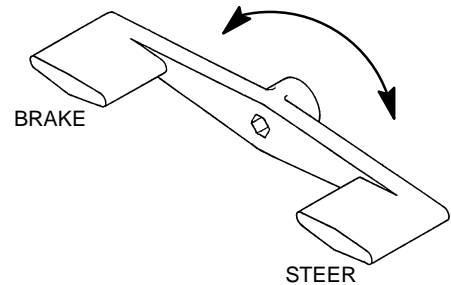

SIDERAIL OPERATION

To engage siderails: Pull up siderail (F) and raise to full up position so that the latch engages.

To disengage siderails: Pull up on latch and guide siderail to the full down position.

CAUTION

Do not allow siderail to lower on its own.

NOTE

If siderail latch is difficult to release, push siderail toward foot end to relieve the pressure on the latch.

FOWLER OPERATION

Squeeze handle (H), for pneumatic assist in lifting the Fowler to the desired height. Remove hand(s) from handle when desired height is achieved. To lower, squeeze handle (H) and push down until the Fowler reaches the desired height.

I.V. POLE OPERATION

1. Remove the I.V. pole (J) from the storage clips on the base hood and insert it into one of the four receptacles on the corners of the litter frame.
2. To raise the height of the pole, turn the knob on the bottom section of the pole counterclockwise and pull up on the top section to raise it to the desired height. Turn the knob clockwise to lock the top section of the pole in place.

Preventative Maintenance

CLEANING

1. Hand wash all surfaces of the bed with warm water and mild detergent. Dry thoroughly.
2. Clean Velcro **AFTER EACH USE**. Saturate Velcro with disinfectant and allow disinfectant to evaporate. (Appropriate disinfectant for nylon Velcro should be determined by the hospital.)

NOTE

Quaternary Germicidal Disinfectants, used as directed, and/or Chlorine Bleach products, typically 5.25% Sodium Hypochlorite in dilutions ranging between 1 part bleach to 100 parts water, and 2 parts bleach to 100 parts water are not considered mild detergents. These products are corrosive in nature and may cause damage to your stretcher if used improperly. If these types of products are used to clean Stryker patient handling equipment, measures must be taken to insure the stretchers are rinsed with clean water and thoroughly dried following cleaning. Failure to properly rinse and dry the stretchers will leave a corrosive residue on the surface of the stretcher, possibly causing premature corrosion of critical components. Failure to follow the above directions when using these types of cleaners may void this product's warranty.

BIANNUAL CHECKLIST

- _____ All fasteners secure
- _____ Siderails move and latch properly
- _____ All casters lock with brake pedal engaged
- _____ Fifth Wheel functioning properly
- _____ All casters secure and swivel properly
- _____ I.V. pole intact and operating properly
- _____ Fowler operates properly
- _____ Trendelenberg/Reverse Trendelenberg operating properly
- _____ No rips or cracks in mattress cover
- _____ No leaks at hydraulic connections
- _____ Hydraulic jacks holding properly
- _____ Hydraulic drop rate set properly
- _____ Hydraulic oil level sufficient

Serial No. _____

Completed By: _____

Date: _____

Service Information

FIELD REPLACEMENT PARTS

<u>PART NAME</u>	<u>PART NUMBER</u>
Fowler Gas Cylinder	946-31-20
Hydraulic Jack, Foot (Control) End	926-20-100
Hydraulic Jack, Head (Non-Control) End	926-21-119
Brake/Steer Pedal	1075-2-203
Caster	1075-2-214
Fifth Wheel Caster	1075-2-215
Panel Fastener	1075-2-208
Plastic Base Hood, Top	1075-2-209
Plastic Base Hood, Bottom	1075-2-210
Release Pedal, Right	1075-2-204
Release Pedal, Left	1075-2-205
Release Pedal Return Spring	1075-2-202
Pedal Pad	946-1-153
Trendelenberg Label	946-1-61
Reverse Trendelenberg Label	946-1-62
Lift Label	946-1-63
Siderail Latch	1075-1-102
Siderail Assembly	1075-1-103
Corner Bumper Roller	1075-1-104
Large "Cap" Retainer	1075-2-211
Small "Cap" Retainer	1075-2-212
Foot End Fiberresin Skin	1075-1-106
Head End Fiberresin Skin	1075-1-107
I.V. Storage Clip	1075-2-213
Mattress Assembly	1075-38-105
Mattress Cover	1075-38-104
I.V. Pole	390-25
Footboard	741-29-100
Footboard/Chartholder	741-34-100
Body Restraint Straps	390-19
Ankle Restraint Straps	946-43-100
Wrist Restraint Straps	946-44-100

Warranty

Limited Warranty:

Stryker Medical Division, a division of Stryker Corporation, warrants to the original purchaser that its products should be free from defects in material and workmanship for a period of one (1) year after date of delivery. Stryker's obligation under this warranty is expressly limited to supplying replacement parts and labor for, or replacing, at its option, any product which is, in the sole discretion of Stryker, found to be defective. Stryker warrants to the original purchaser that the frame and welds on its beds will be free from structural defects for as long as the original purchaser owns the bed. If requested by Stryker, products or parts for which a warranty claim is made shall be returned prepaid to Stryker's factory. Any improper use or any alteration or repair by others in such manner as in Stryker's judgement affects the product materially and adversely shall void this warranty. No employee or representative of Stryker is authorized to change this warranty in any way.

This statement constitutes Stryker's entire warranty with respect to the aforesaid equipment. STRYKER MAKES NO OTHER WARRANTY OR REPRESENTATION, EITHER EXPRESSED OR IMPLIED, EXCEPT AS SET FORTH HEREIN. THERE IS NO WARRANTY OF MERCHANTABILITY AND THERE ARE NO WARRANTIES OF FITNESS FOR ANY PARTICULAR PURPOSE. IN NO EVENT SHALL STRYKER BE LIABLE HEREUNDER FOR INCIDENTAL OR CONSEQUENTIAL DAMAGES ARISING FROM OR IN ANY MANNER RELATED TO SALES OR USE OF ANY SUCH EQUIPMENT.

To Obtain Parts and Service:

Stryker products are supported by a nationwide network of dedicated Stryker Field Service Representatives. These representatives are factory trained, available locally, and carry a substantial spare parts inventory to minimize repair time. Simply call your local representative, or call Stryker Customer Service at (800) 327-0770.

Supplemental Warranty Coverage:

Stryker has developed a comprehensive program of extended warranty options designed to keep your equipment operating at peak performance at the same time it eliminates unexpected costs. We recommend that these programs be activated *before* the expiration of the new product warranty to eliminate the potential of additional equipment upgrade charges. Stryker offers the following Supplemental Warranties:

Extended (Parts and Labor)

- All replacement parts (excluding mattresses and consumable items)
- Labor and travel for *all* scheduled and unscheduled calls
- Annual Preventive Maintenance Inspections and repairs
- JCAHO paperwork for preventive maintenance
- Priority Emergency Service

Standard (Labor Only):

- Labor and travel for *all* scheduled and unscheduled calls
- Annual Preventive Maintenance Inspections and repairs
- JCAHO paperwork for preventive maintenance
- Priority Emergency Service

Basic (Parts Only):

- All replacement parts (excluding mattresses and consumable items)
- Priority Emergency Service

Please call your local representative, or call (800) 327-0770 for further information

Warranty

Return Authorization:

Merchandise cannot be returned without approval from the Stryker Customer Service Department. An authorization number will be provided which must be printed on the returned merchandise. Stryker reserves the right to charge shipping and restocking fees on returned items.

SPECIAL, MODIFIED, OR DISCONTINUED ITEMS NOT SUBJECT TO RETURN.

Damaged Merchandise:

ICC Regulations require that claims for damaged merchandise must be made with the carrier within fifteen (15) days of receipt of merchandise. **DO NOT ACCEPT DAMAGED SHIPMENTS UNLESS SUCH DAMAGE IS NOTED ON THE DELIVERY RECEIPT AT THE TIME OF RECEIPT.** Upon prompt notification, Stryker will file a freight claim with the appropriate carrier for damages incurred. Claim will be limited in amount to the actual replacement cost. In the event that this information is not received by Stryker within the fifteen (15) day period following the delivery of the merchandise, or the damage was not noted on the delivery receipt at the time of receipt, the customer will be responsible for payment of the original invoice in full.

Claims for any short shipment must be made within thirty (30) days of invoice.

International Warranty Clause:

This warranty reflects U.S. domestic policy. Warranty outside the U.S. may vary by country. Please contact your local Stryker Medical representative for additional information.



6300 Sprinkle Road, Kalamazoo, MI 49001-9799

(800) 327-0770

DH 6/93 1075-1-101 REV__