


stryker®


Parts Manual


Medical


Fluoroscopy Stretcher
Model 1080



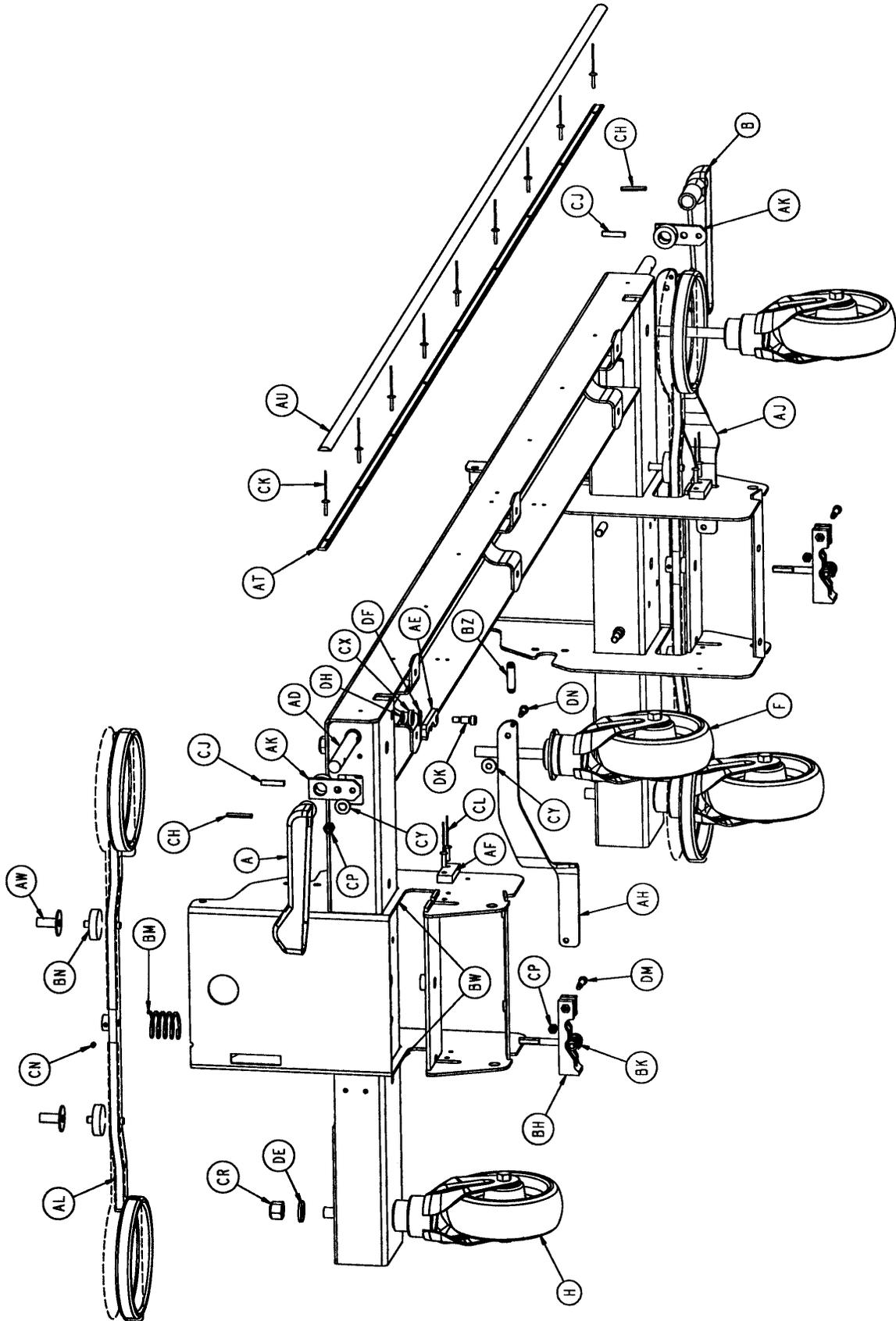

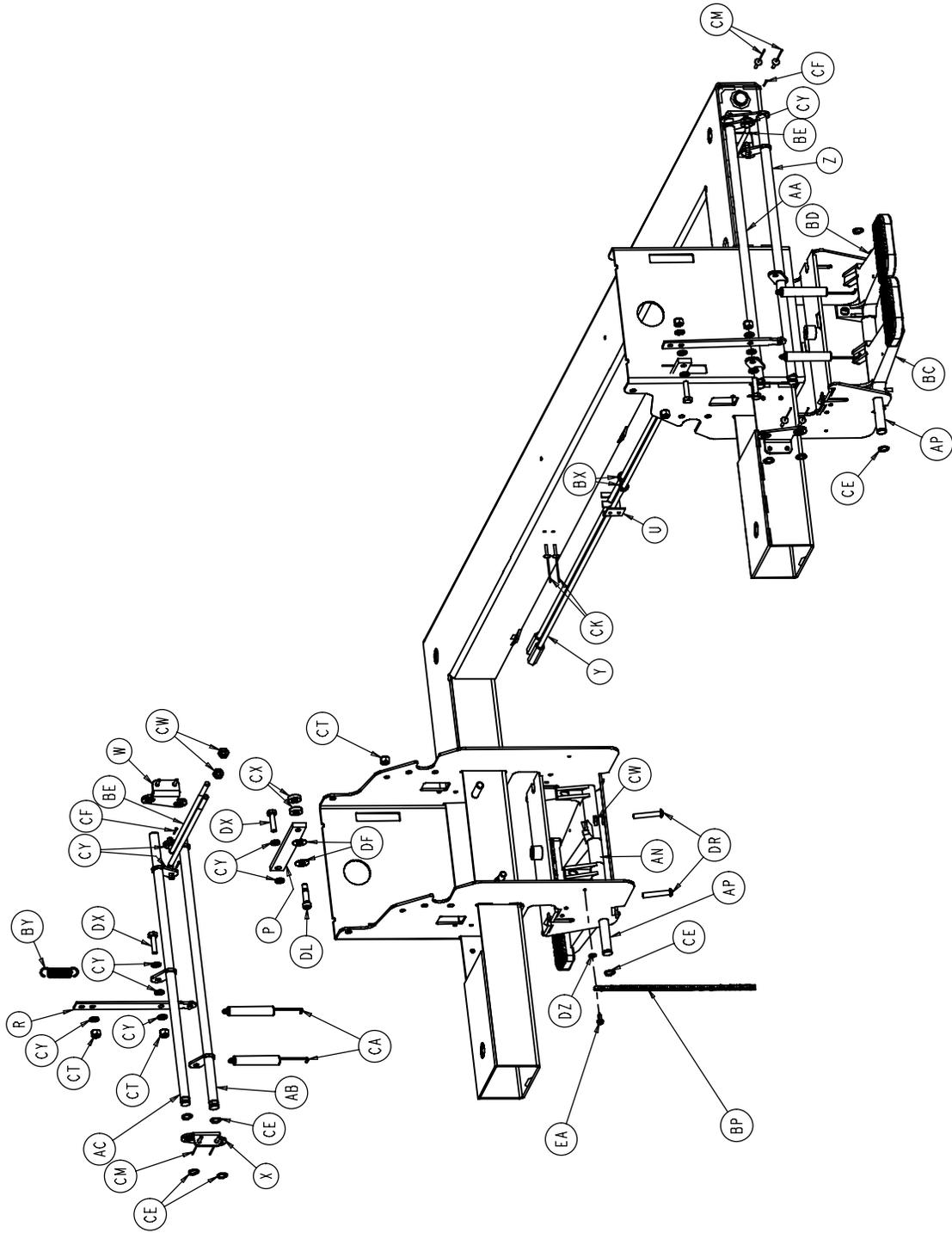

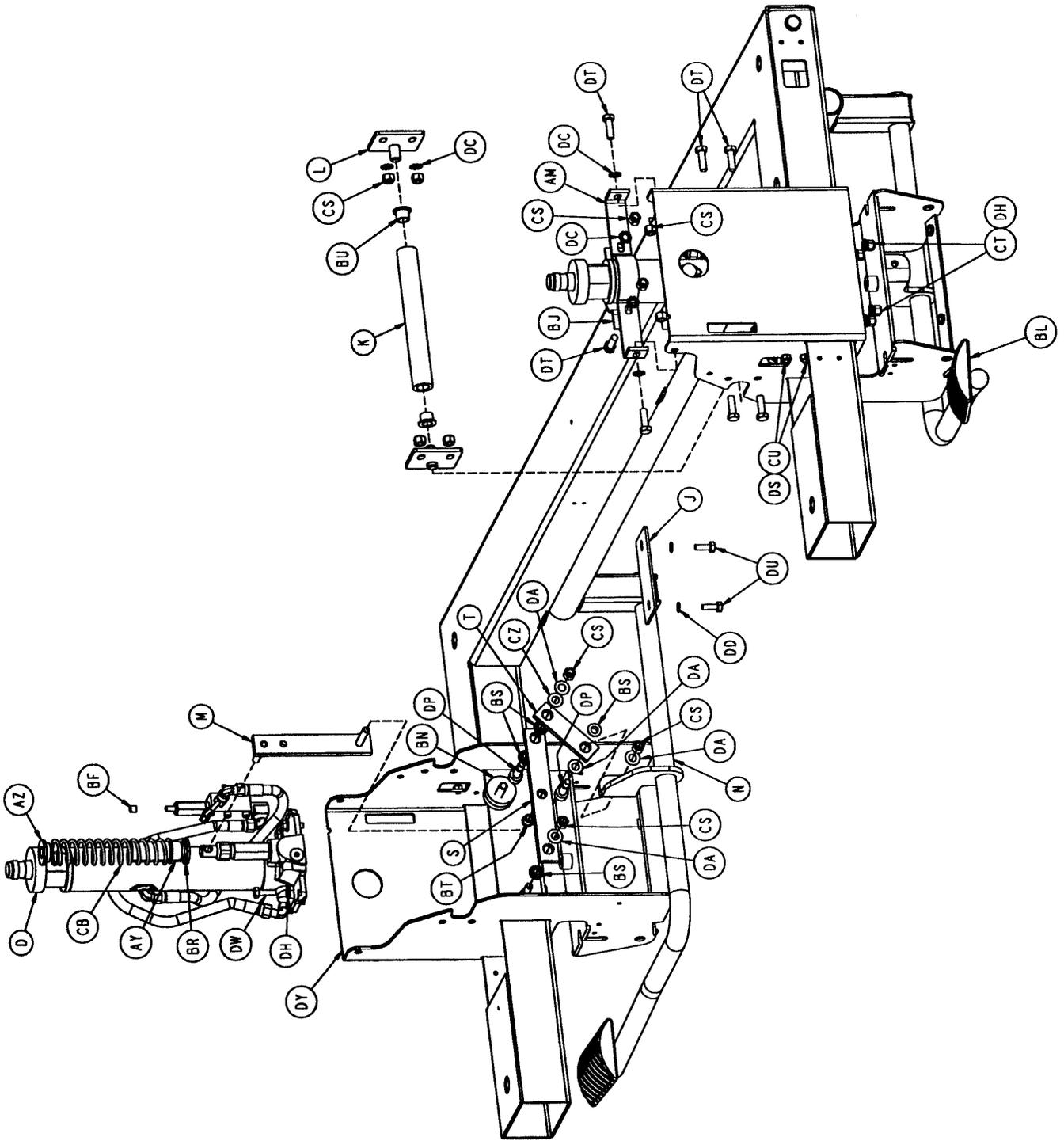


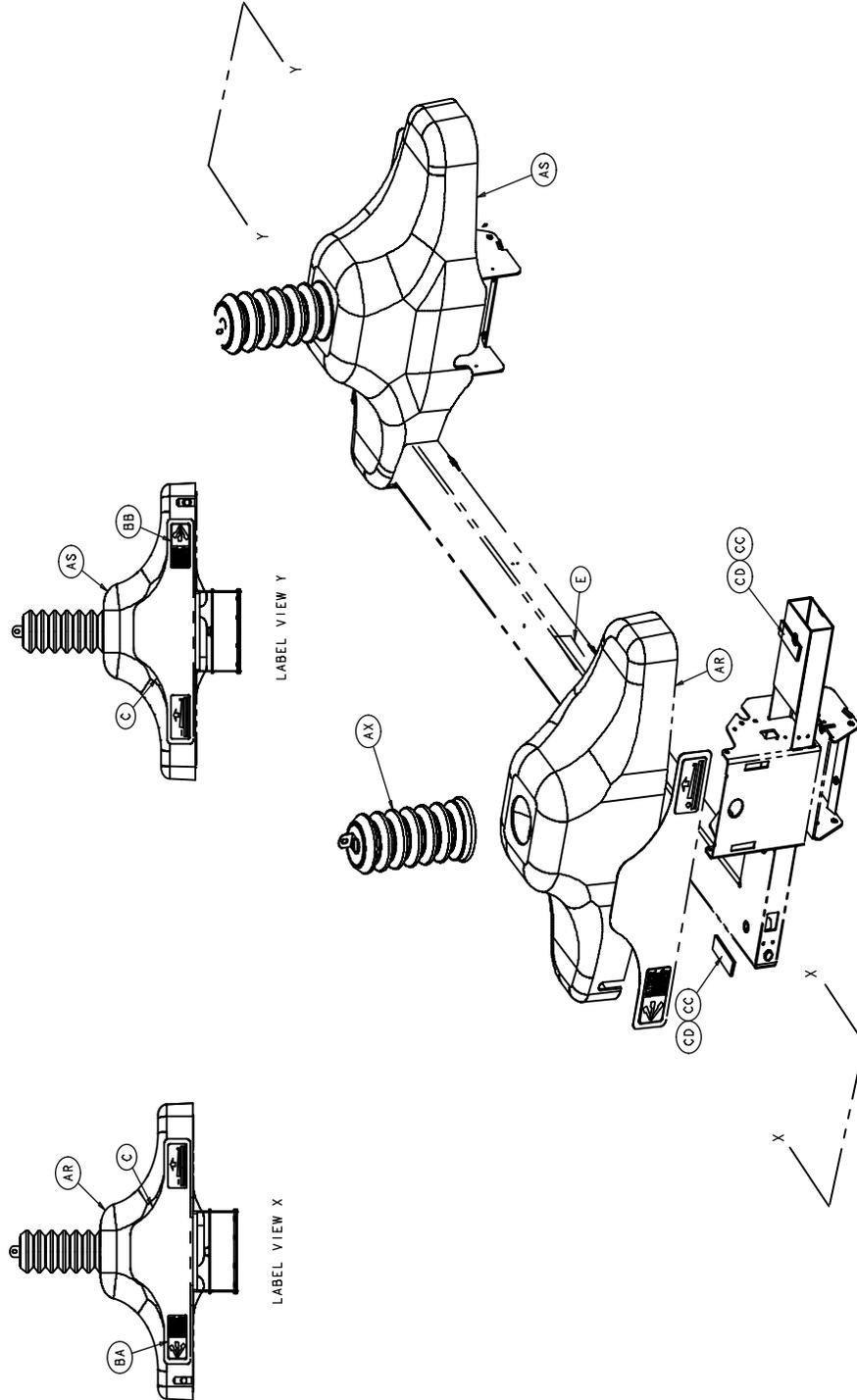

1080-91-10


A	1210-1-346	1	BL	715-1-126	2
B	1210-1-345	1	BM	715-1-94	2
C	1080-90-1	2	BN	715-1-11	8
D	1080-70-10	2	BP	3001-200-52	1
E	1080-30-40	1	BR	390-1-67	2
F	1080-5-11	1	BS	81-240	8
H	1080-5-10	3	BT	81-222	2
J	1080-4-94	3	BU	81-213	4
K	1080-4-88	2	BW	58-56	.75'
L	1080-4-55	4	BX	52-245	4
M	1080-4-54	2	BY	38-356	2
N	1080-4-50	1	BZ	38-355	1
P	1080-3-94	2	CA	38-353	4
R	1080-3-93	2	CB	38-246	2
S	1080-3-82	2	CC	29-9	8
T	1080-3-81	2	CD	29-7	8
U	1080-3-80	2	CE	28-97	12
W	1080-3-78	2	CF	27-4	4
X	1080-3-77	2	CH	26-196	2
Y	1080-3-57	2	CJ	26-13	2
Z	1080-3-53	1	CK	25-103	14
AA	1080-3-52	1	CL	25-86	4
AB	1080-3-51	1	CM	25-79	8
AC	1080-3-50	1	CN	21-50	2
AD	1080-2-98	1	CP	16-59	4
AE	1080-2-94	1	CR	16-57	4
AF	1080-2-93	2	CS	16-36	20
AH	1080-2-92	1	CT	16-28	14
AJ	1080-2-91	1	CU	16-3	4
AK	715-1-165	2	CW	15-11	8
AL	1080-2-50	2	CX	14-70	4
AM	1080-1-89	2	CY	14-46	20
AN	1080-1-88	2	CZ	14-21	2
AP	1080-1-87	2	DA	14-20	8
AR	1080-1-85	1	DC	13-38	16
AS	1080-1-84	1	DD	13-10	6
AT	1080-1-78	1	DE	11-310	4
AU	1010-700-75	1	DF	11-179	7
AW	946-1-116	4	DH	11-63	17
AX	926-1-48	2	DK	8-49	1
AY	763-1-16	2	DL	8-27	2
AZ	715-1-133	2	DM	8-21	2
BA	721-91-45	1	DN	8-17	2
BB	721-91-44	1	DP	8-4	4
BC	716-1-294	2	DR	4-135	4
BD	716-1-293	2	DS	4-85	4
BE	716-1-75	4	DT	3-85	16
BF	715-1-333	2	DU	3-50	6
BH	715-301-221	2	DW	3-15	8
BJ	715-1-193	2	DX	3-3	4
BK	715-1-150	2	DY	1080-1-50	1
			DZ	13-18	1
			EA	3-364	1

1080-70-10

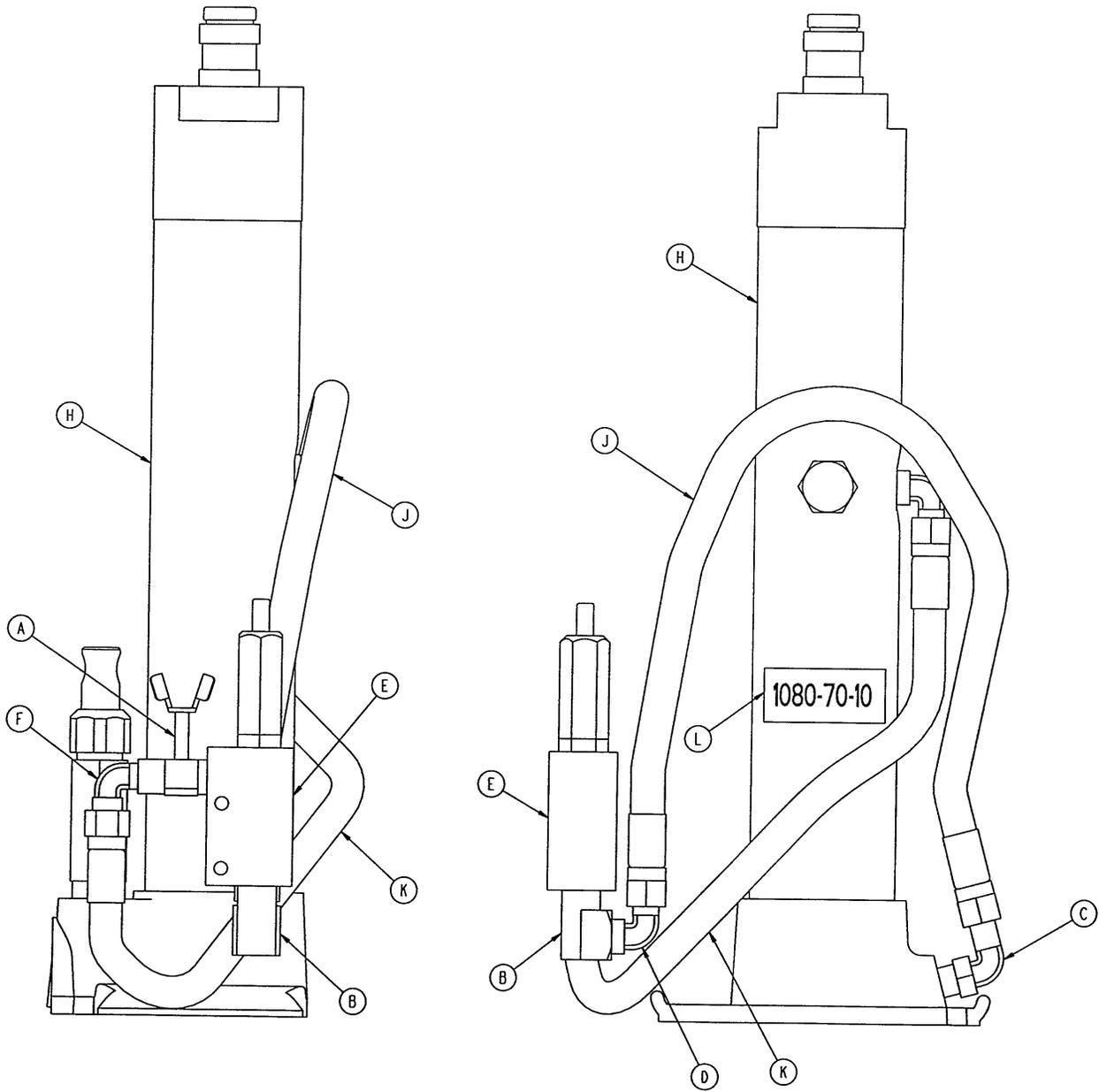


A	48-13	1
B	48-1271	
C	48-146	1
D	390-2-147	1
E	962-1-59	1
F	988-70-456	1
H	1080-70-50	1
J	1080-70-98	1
K	1080-70-99	1
L	1080-90-2	1

1080-70-50

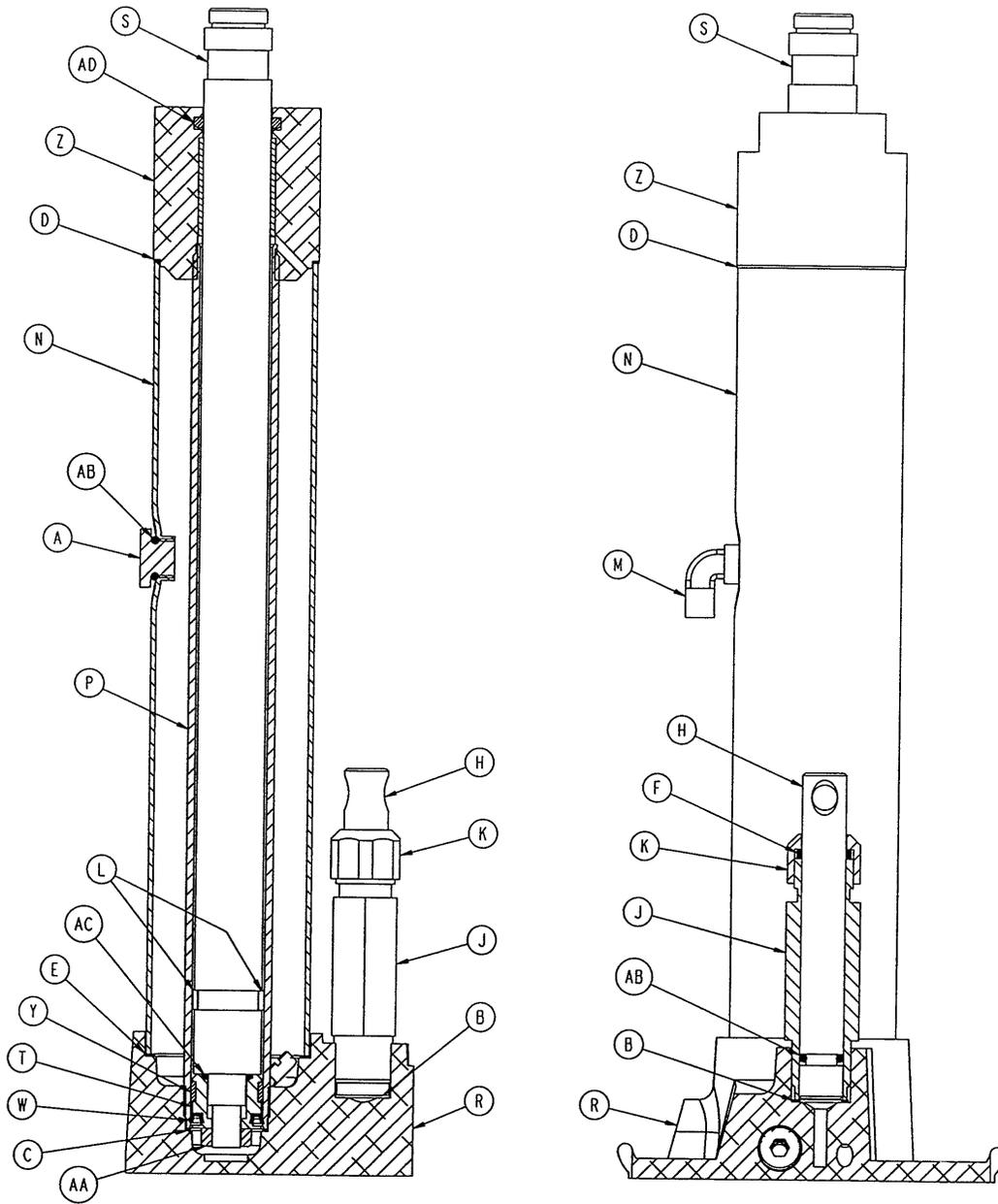


A	388-1-38	1	P	921-27-12	1
B	390-1-237	1	R	926-1-295	1
C	390-1-238	1	S	926-20-127	1
D	390-1-243	1	T	926-20-160	1
E	390-1-244	1	W	926-20-161	1
F	390-2-128	1	Y	926-20-162	1
H	390-2-135	1	Z	938-1-31	1
J	390-2-136	1	AA	15-13	1
K	390-2-137	1	AB	45-110	2
L	390-2-139	2	AC	45-14	1
M	390-2-147	1	AD	45-904	1
N	921-27-11	1			

926-1-295

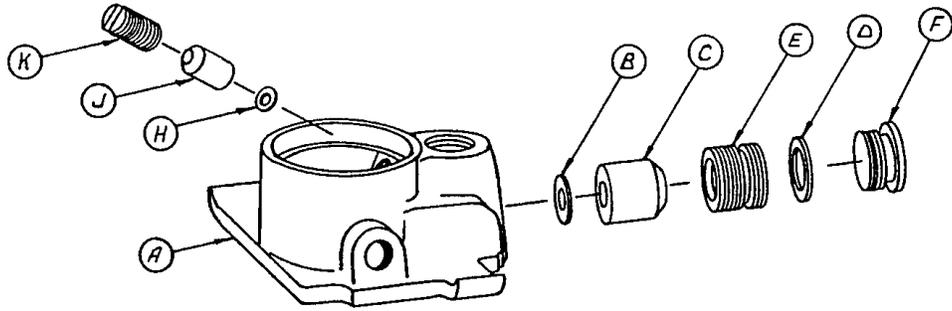


A	946-1-291	1
B	926-20-154	1
C	926-20-153	1
D	926-20-156	1
E	926-20-159	1
F	926-20-157	1
H	926-20-155	1
J	926-20-152	1
K	926-20-158	1

1080-5-10

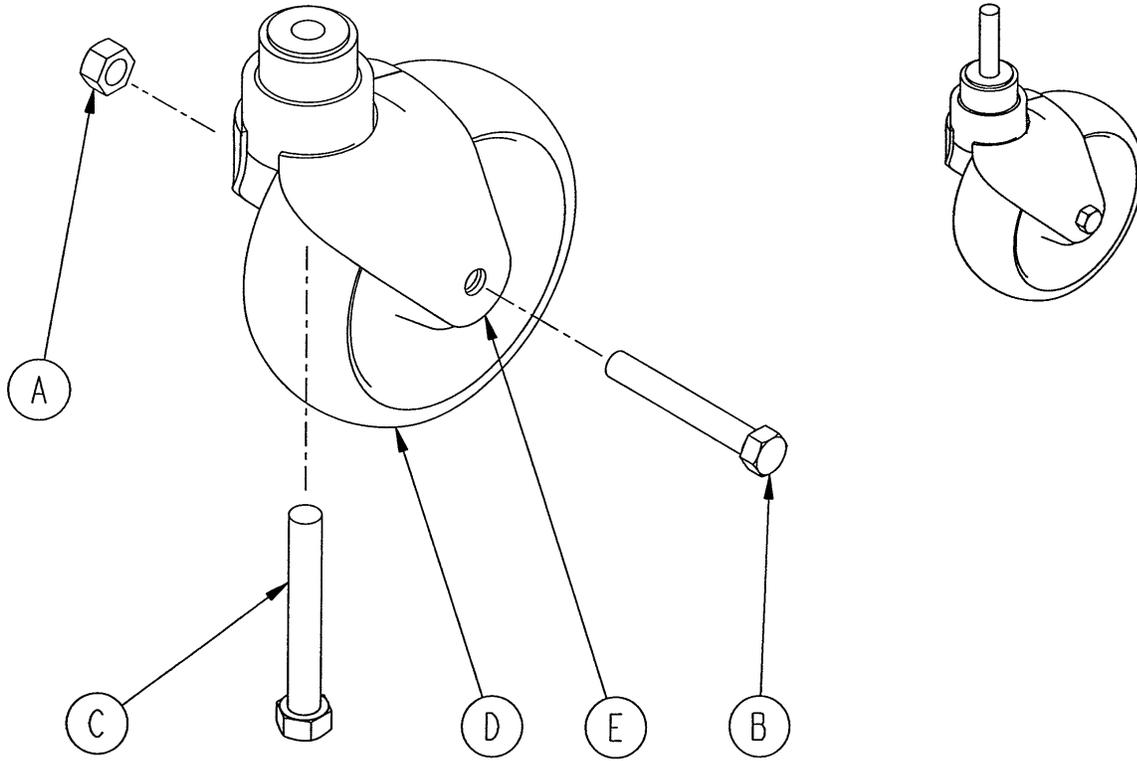


A	16-60	1
B	3-205	1
C	3-215	1
D	5050-2-10	1
E	5050-2-30	1

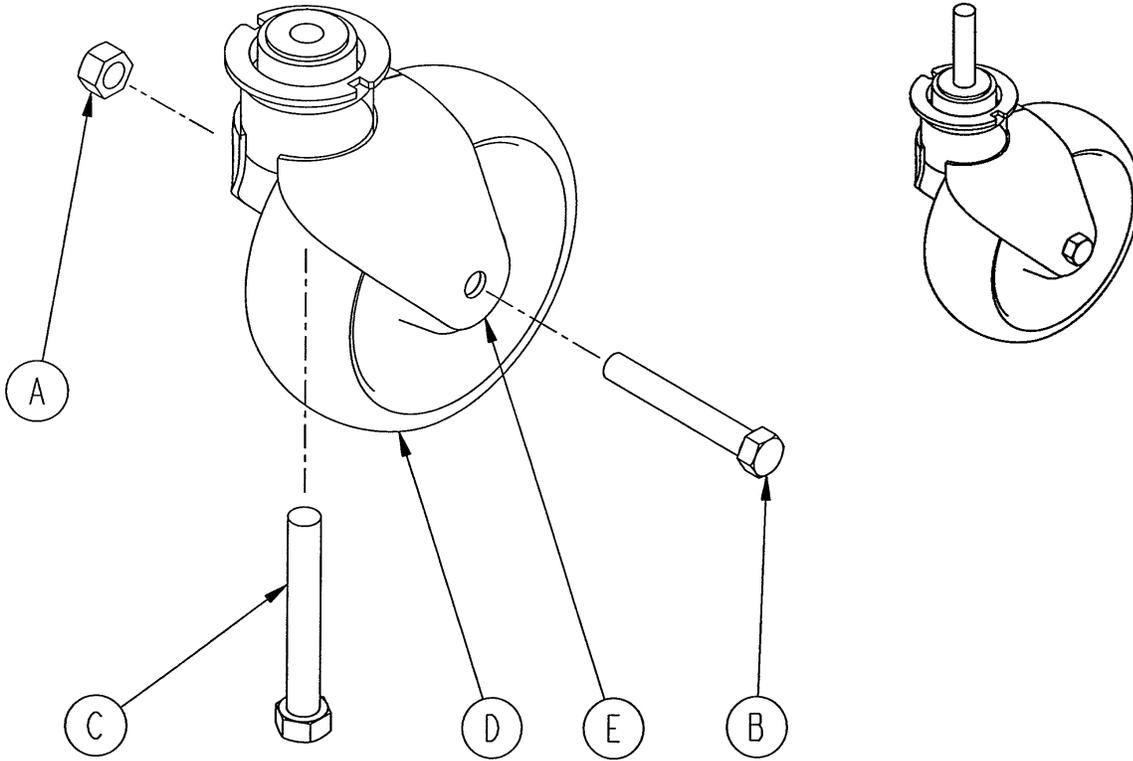


A	16-60	1
B	3-205	1
C	3-215	1
D	5050-2-10	1
E	5050-2-31	1

5050-2-10


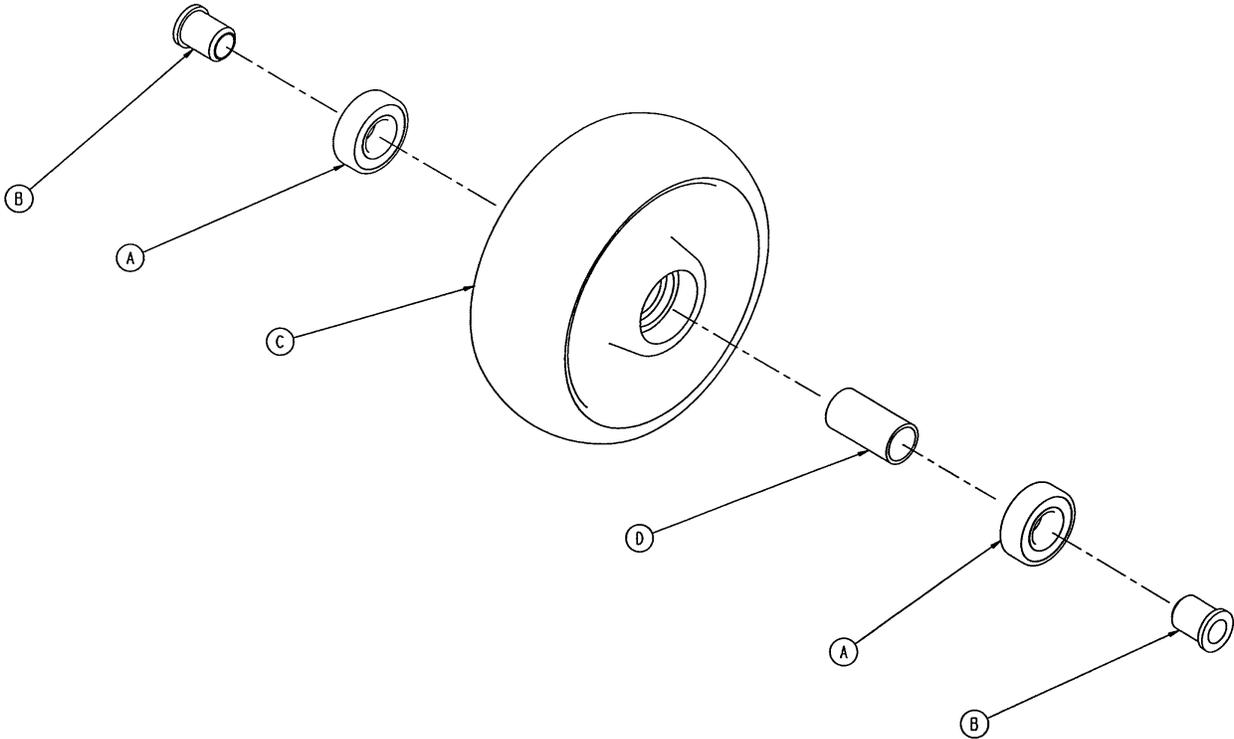


A	81-226	2
B	715-1-255	2
C	5050-2-20	1
D	6060-2-41	1


715-201-150

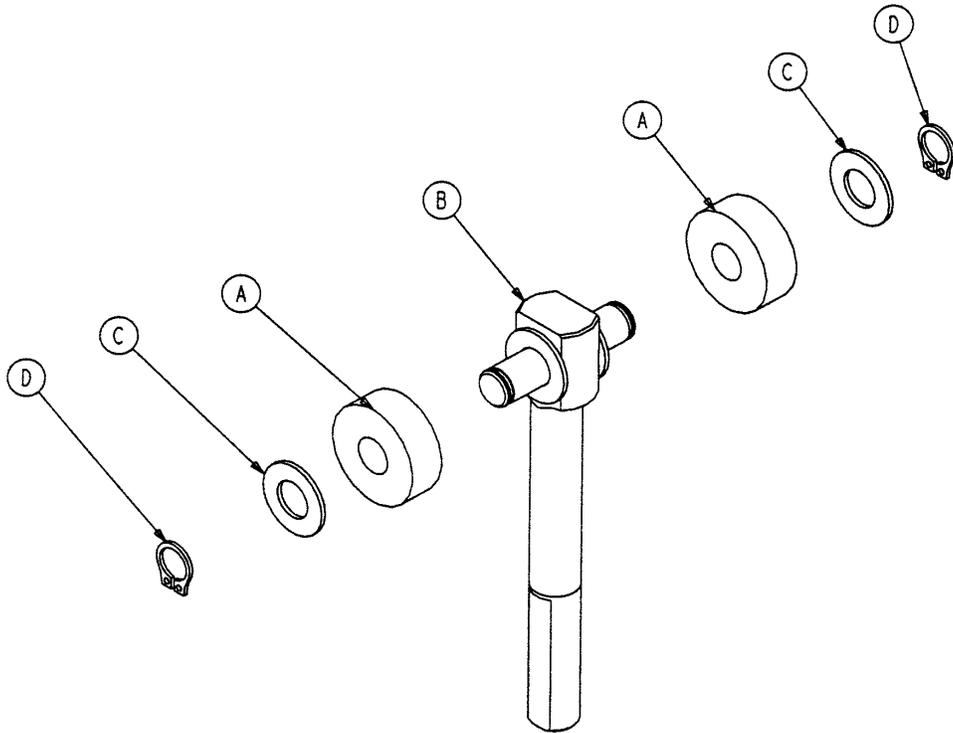

A	715-1-180	2
B	715-201-62	1
C	14-4	4
D	28-8	2

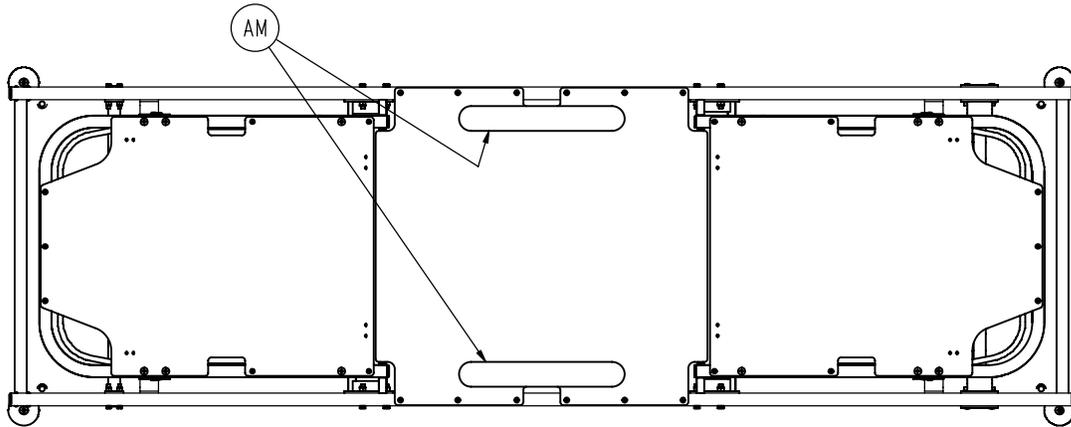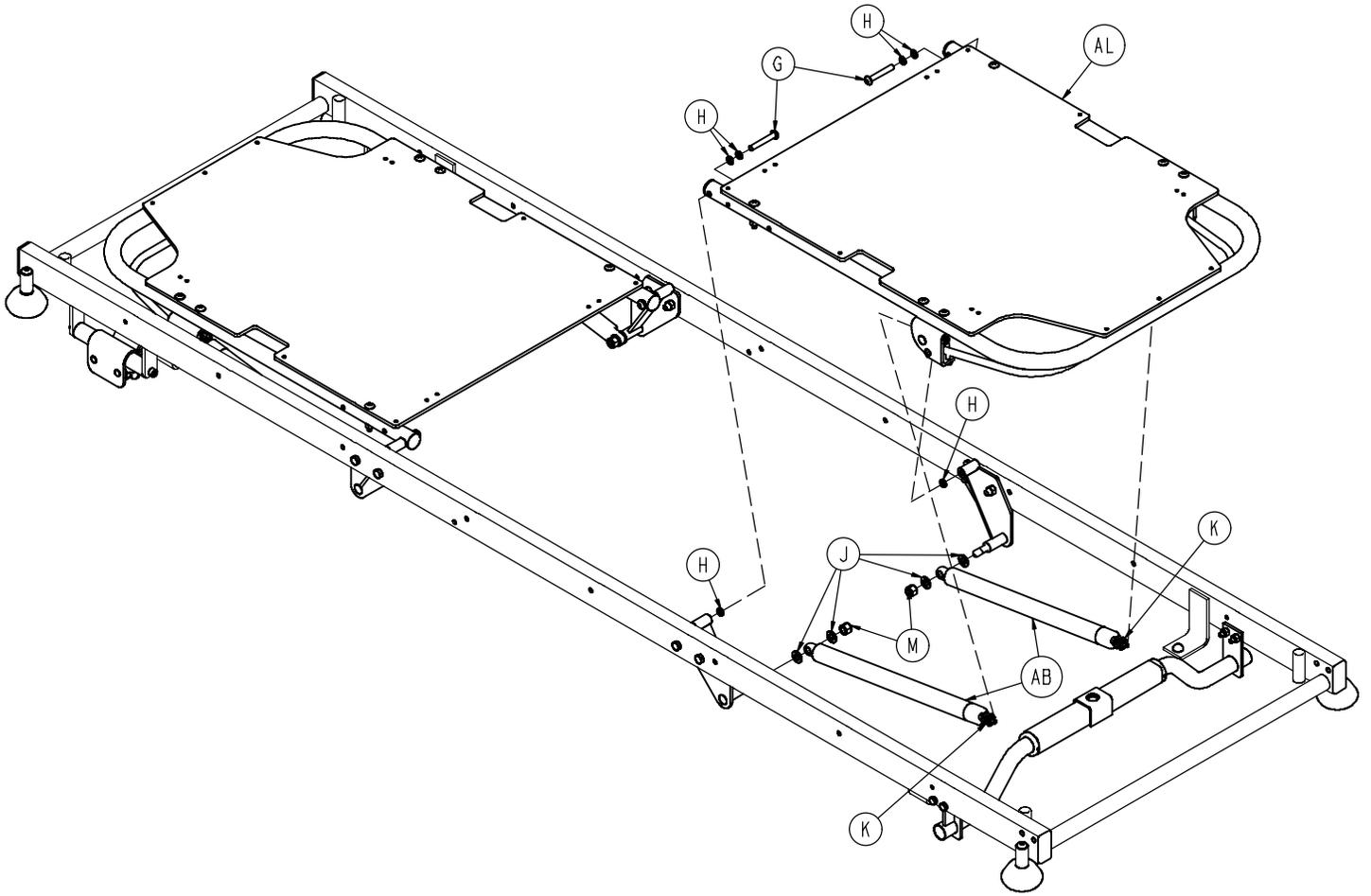


1080-30-10

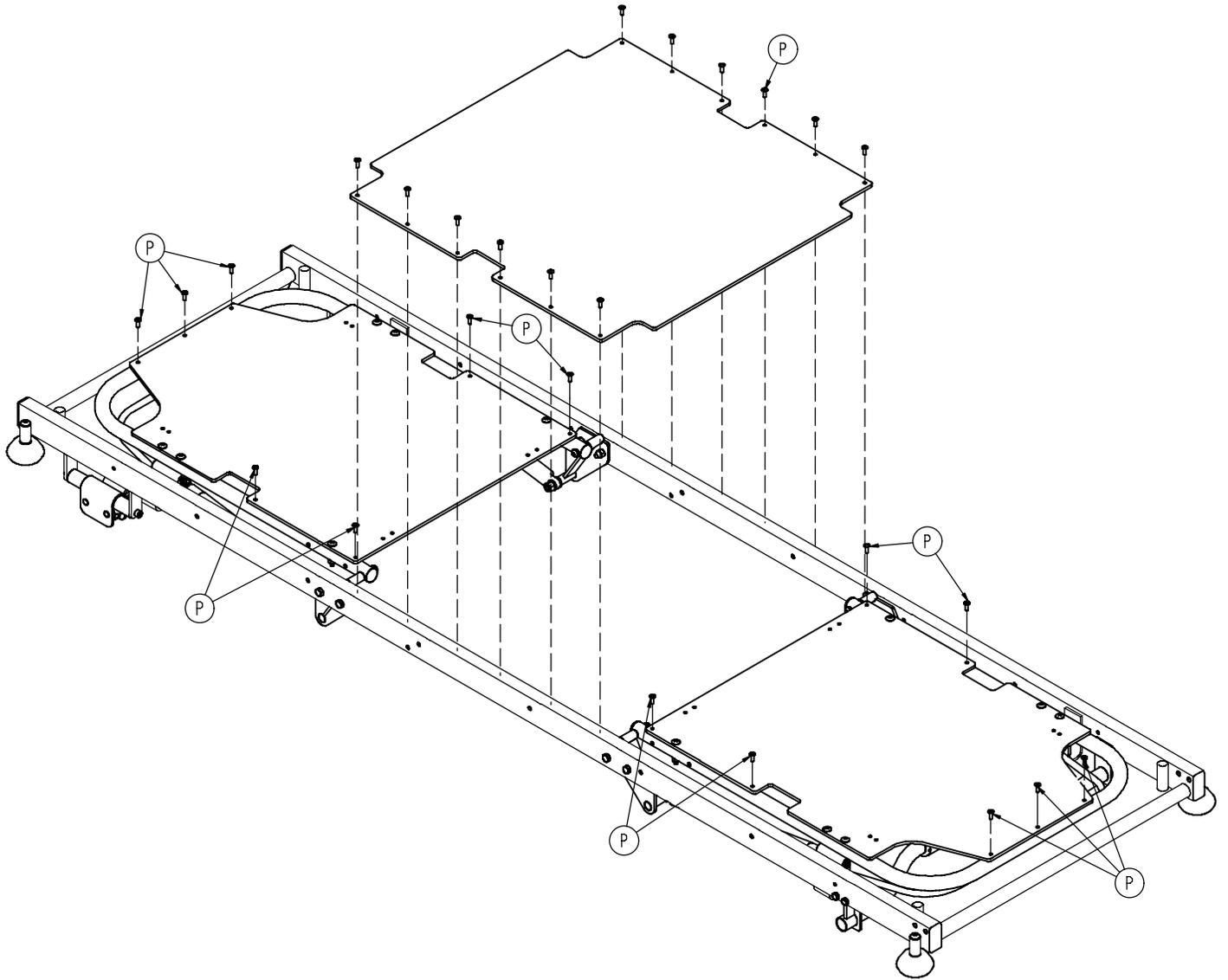


A	3-40	2	U	46-1	2
B	3-68	8	V	56-4	4
C	3-78	4	W	721-31-65	8
D	4-21	4	Y	926-400-142	4
E	4-49	4	AA	938-1-401	4
F	4-182	4	AB	1010-31-78	4
G	4-183	4	AC	1010-32-85	2
H	14-3	12	AD	1020-1-49	2
J	14-21	16	AE	1020-1-59	2
K	15-59	4	AF	1080-30-13	2
L	16-28	4	AH	1080-30-14	2
M	16-35	8	AJ	1080-30-23	2
N	16-36	10	AK	1080-30-24	2
P	25-50	26	AL	1080-30-35	1
R	37-30	2	AM	1080-31-120	2
T	37-206	4	AN	7900-1-102	2

1080-31-120

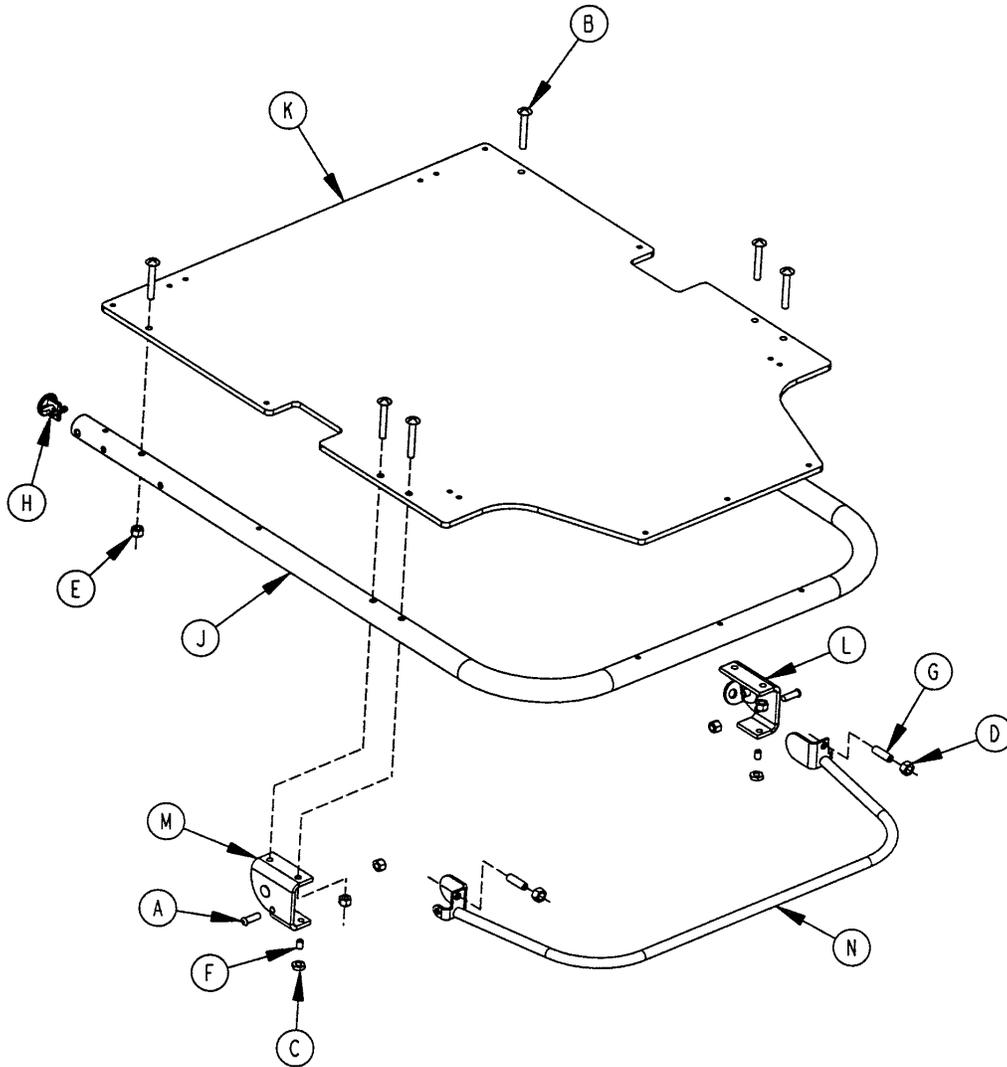


A	4-161	2
B	7-20	6
C	15-37	2
D	15-50	2
E	16-28	8
F	21-125	2
G	21-126	2
H	1001-1-36	2
J	1001-31-13	1
K	1010-231-12	1
L	1210-31-106	1
M	1210-31-107	1
N	1210-31-118	1

1210-31-106 / 1210-31-107

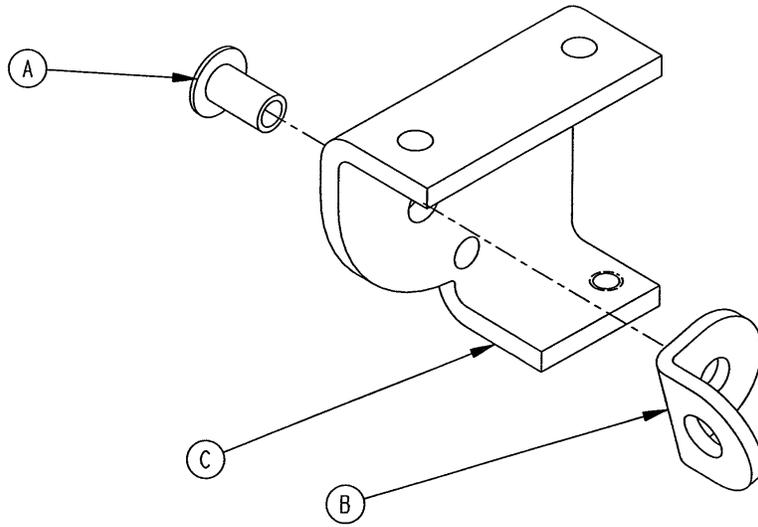


A	25-174	1
B	1210-31-103	1
C	1210-31-104	1

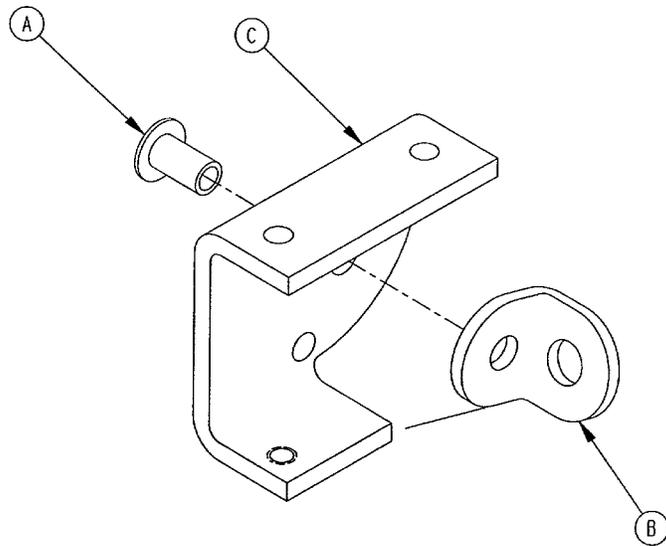


A	25-174	1
B	1210-31-103	1
C	1210-31-105	1

1080-20-10

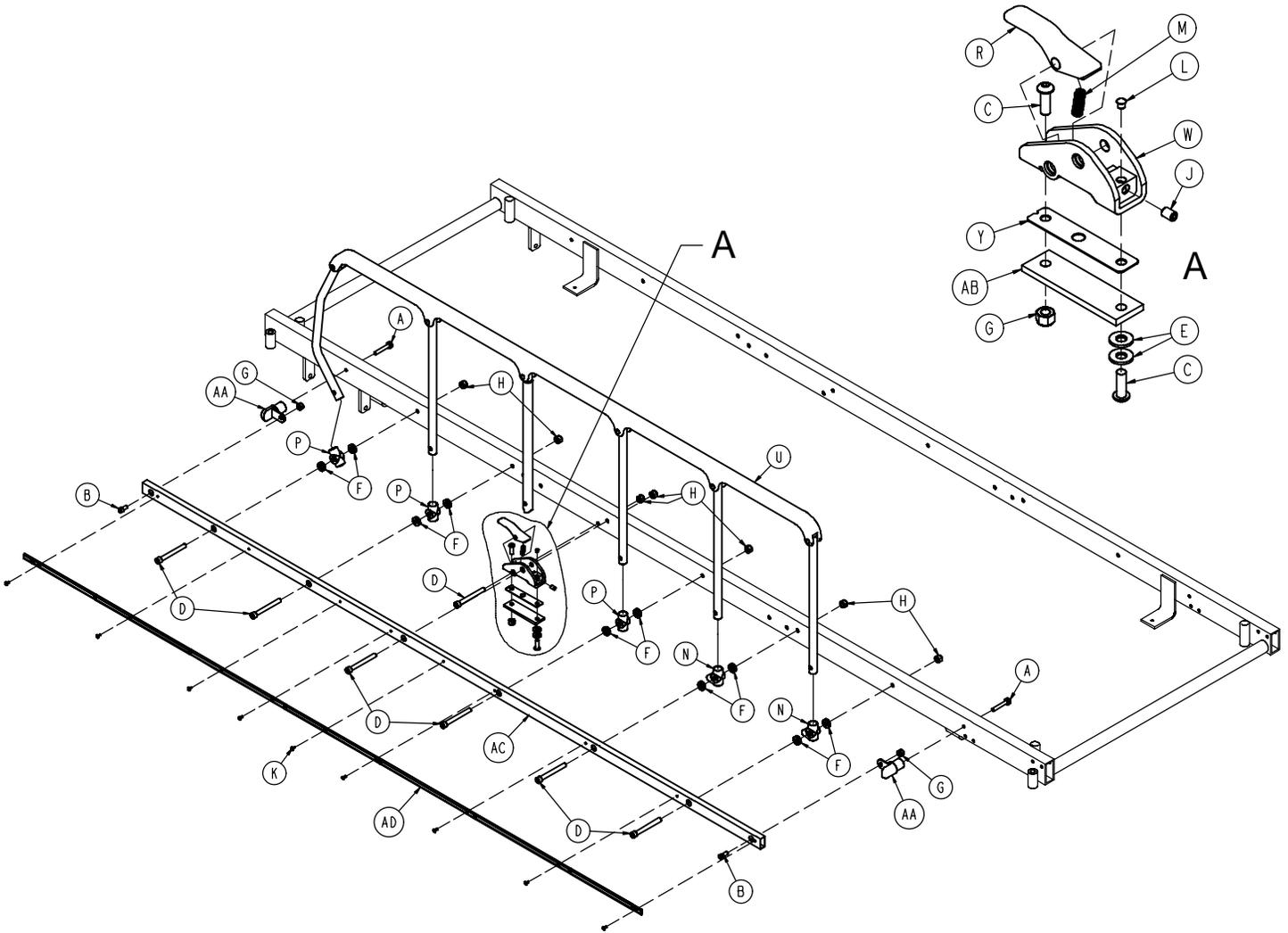


A	3-47	4	N	721-26-68	4
B	4-18	4	P	721-26-69	6
C	4-161	4	R	721-26-74	2
D	4-272	14	T	1010-26-17	1
E	11-2	4	U	1010-26-19	1
F	14-70	20	V	1010-26-80	1
G	16-28	6	W	1010-26-81	1
H	16-36	14	Y	1010-26-103	2
J	21-104	2	AA	1080-30-16	4
K	25-38	18	AB	1080-30-31	2
L	37-74	2	AC	1080-30-32	2
M	38-220	2	AD	1080-30-33	2

1010-26-17 & 1010-26-19

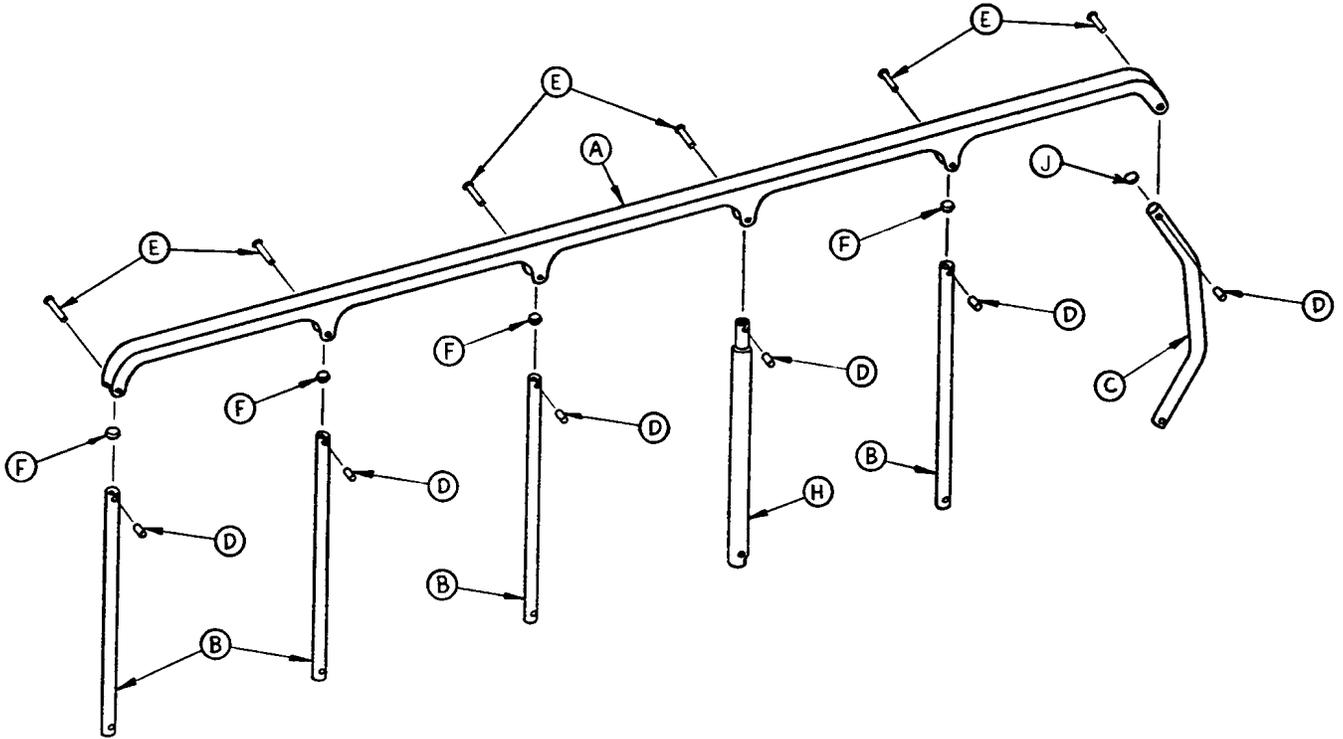


A	1010-26-15	1
B	1010-26-83	4
C	1010-26-84	1
D	1010-26-82	6
E	25-106	6
F	1010-26-10	4
H	1010-26-85	1
J	1010-26-12	1

1080-22-10

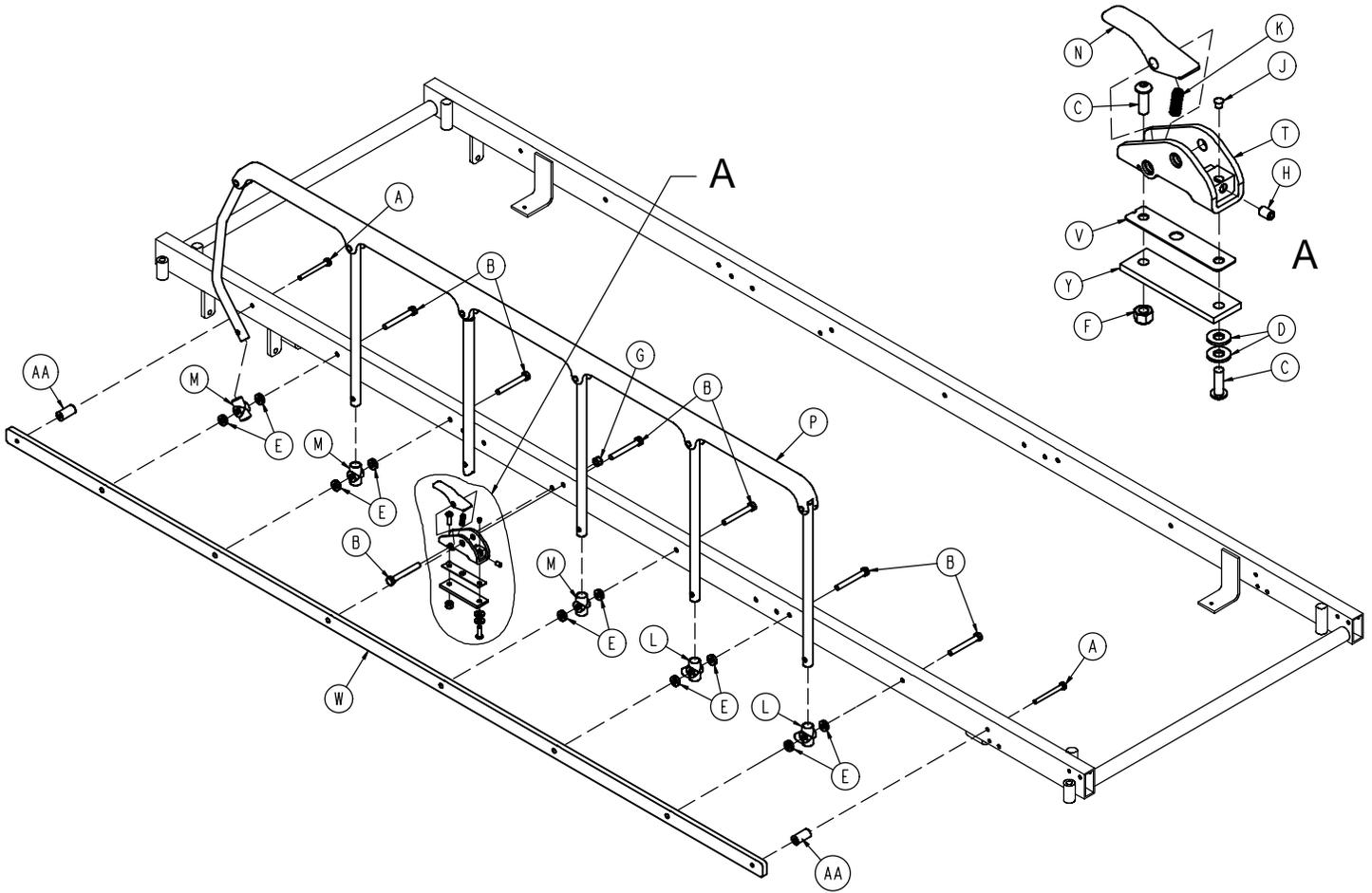


A	3-23	4	M	721-26-69	6
B	3-82	14	N	721-26-74	2
C	4-161	4	P	1010-26-17	1
D	11-2	1	R	1010-26-19	1
E	14-70	20	T	1010-26-80	1
F	16-28	2	U	1010-26-81	1
G	16-36	2	V	1010-26-103	2
H	21-104	2	W	1080-22-15	2
J	37-74	2	Y	1080-30-31	2
K	38-220	2	AA	1080-30-38	4
L	721-26-68	4			

1080-134-10

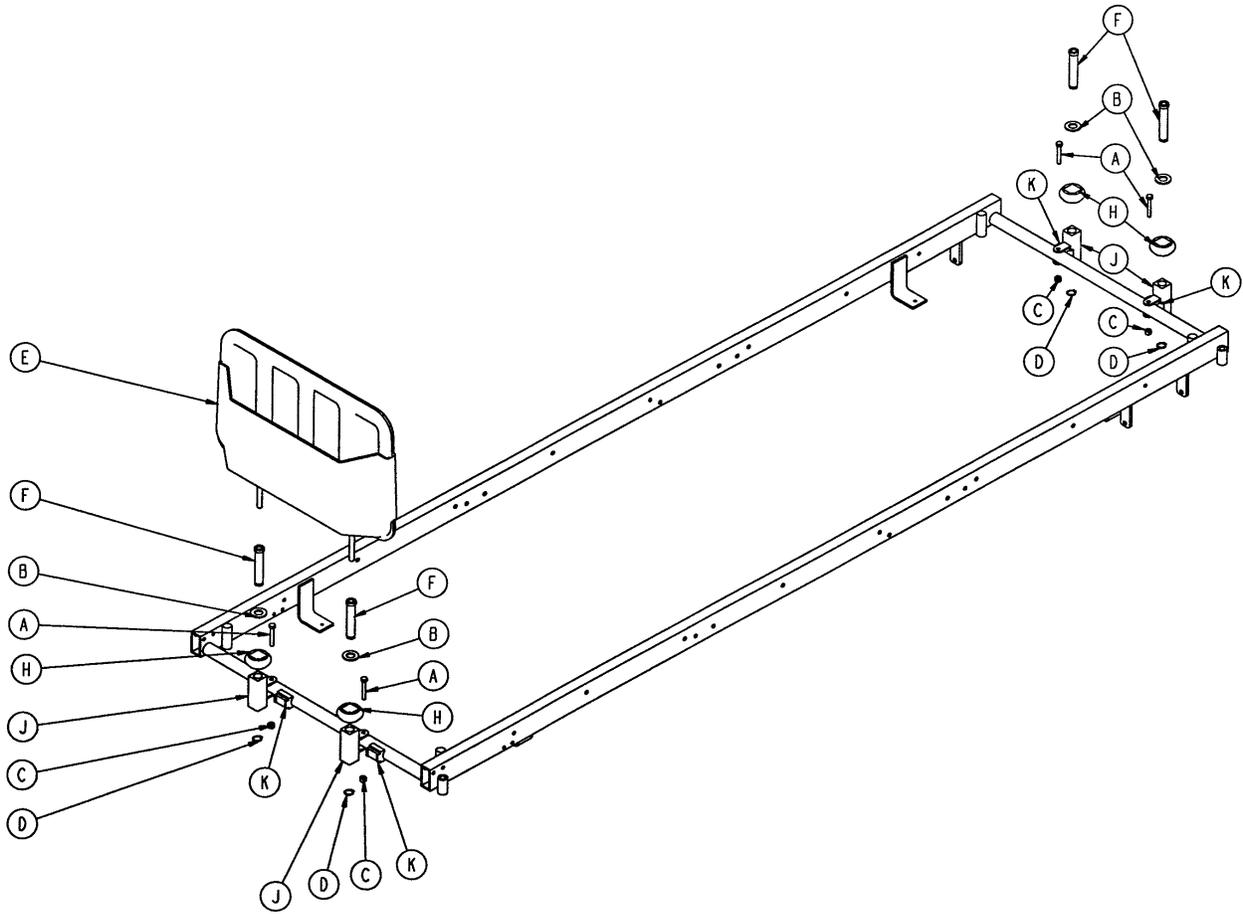


A	3-78	4
B	14-71	4
C	16-28	4
D	28-72	4
E	926-40-100	1
F	1001-40-12	4
H	1010-51-18	4
J	1080-134-12	4
K	1080-134-13	4

1080-110 & 1080-112

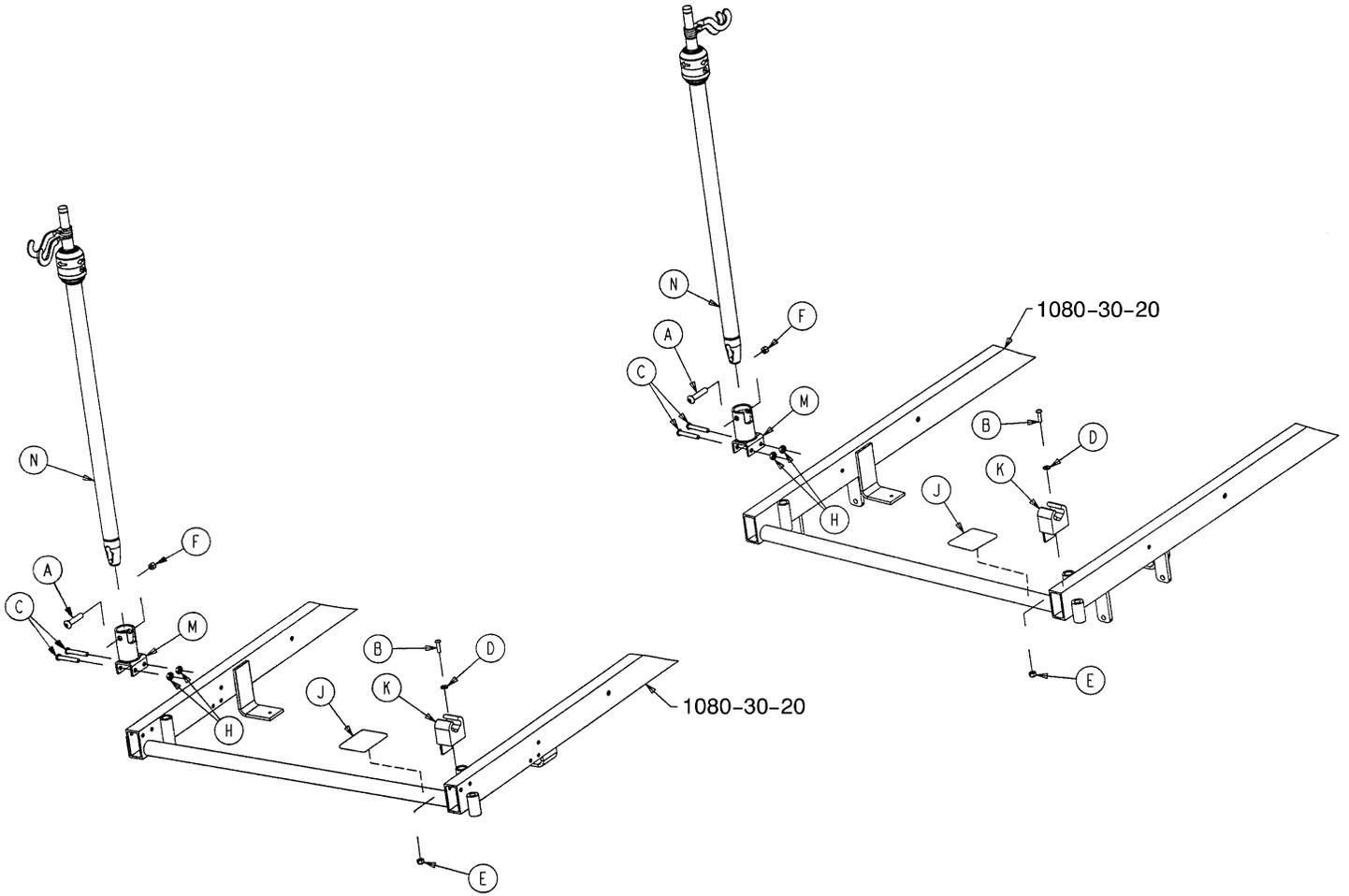


A	4-199	1
B	4-169	1
C	4-135	2
D	11-16	1
E	16-3	1
F	16-36	1
H	16-5	2
J	1001-1-56	1
K	1001-59-12	1
M	1080-110-112	1
N	1211-210-10	1

1211-210-010

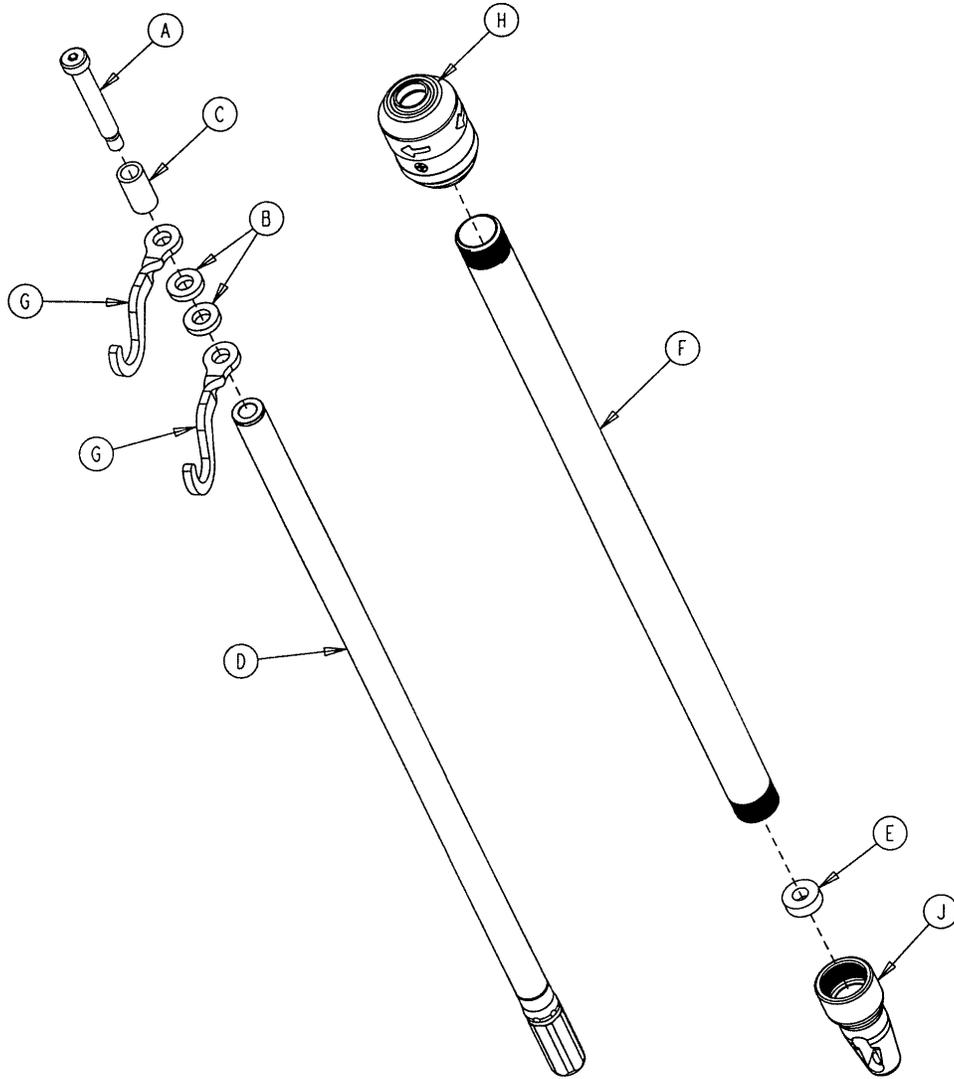


A	0008-031-000	1
B	0052-017-000	2
C	0926-400-162	1
D	1211-210-029	1
E	1001-359-013	1
F	1001-159-028	1
G	1010-059-016	2
H	1211-110-026	1
J	1001-359-112	1

1211-210-026

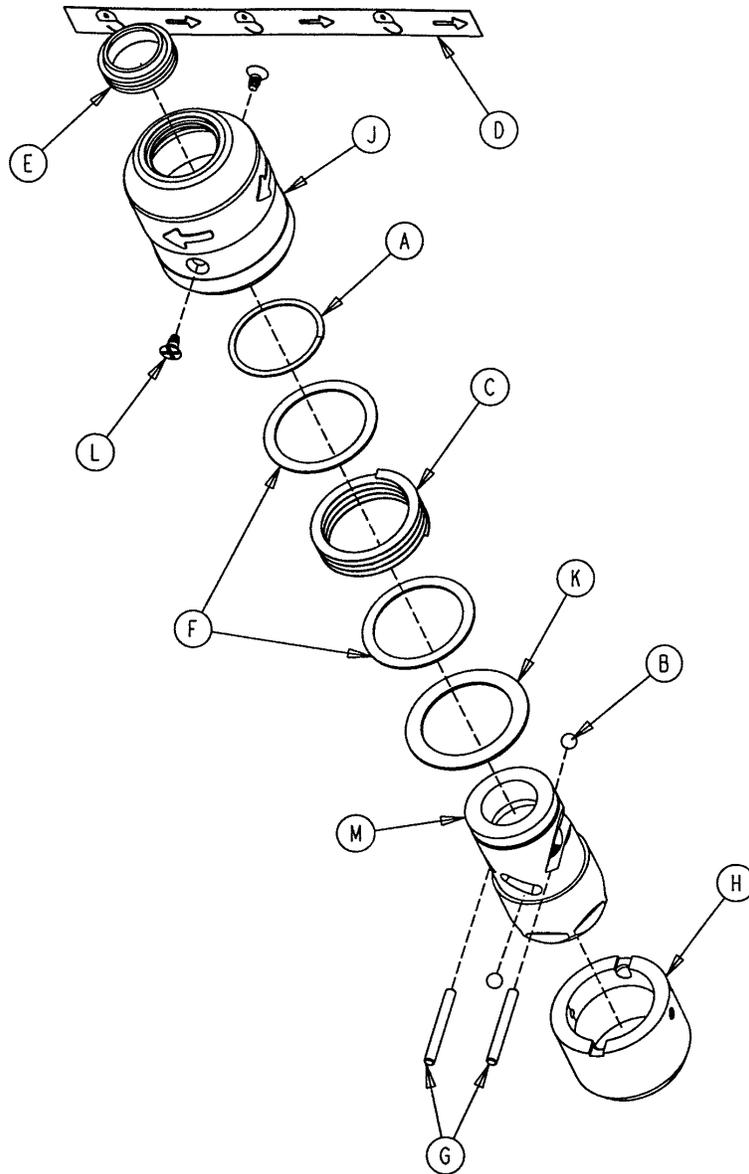


A	0028-167-000	1
B	0031-004-000	2
C	0038-392-000	1
D	1211-091-034	1
E	1211-110-018	1
F	1211-110-020	2
G	1211-110-021	2
H	1211-110-022	1
J	1211-110-024	1
K	1211-110-035	1
L	1211-110-036	2
M	1211-210-023	1

A	0024-023-000	1
B	0390-003-053	1
C	0393-003-043	1
D	0004-496-000	1

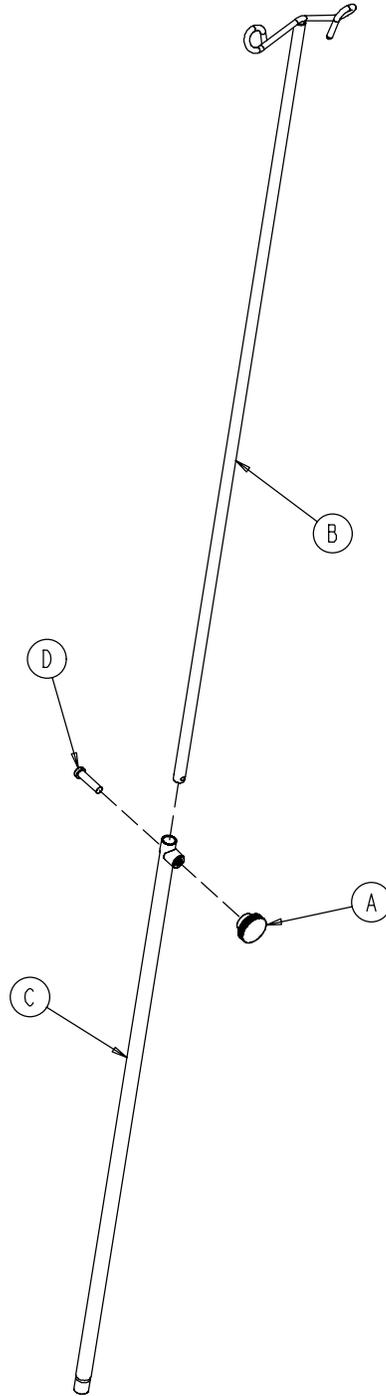


0926-040-100

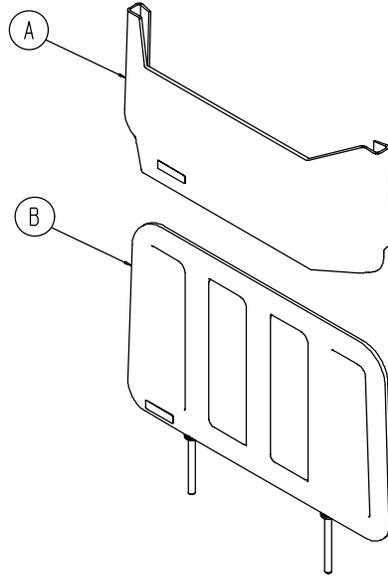

A	0946-028-100	1
B	0946-029-001	1

1020-030-000

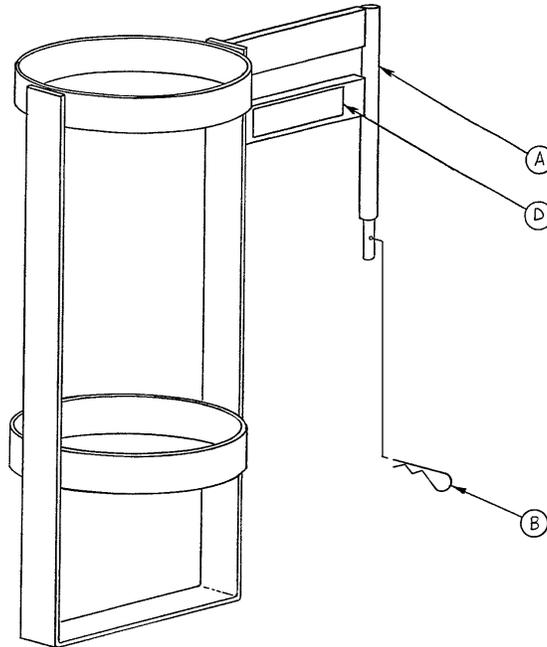

A	1020-030-011	1
B	0027-012-000	1
D	1020-030-017	1

International Addresses



EASTERN EUROPE

Stryker SA - Export Business

Grand-Rue 90
P.O. Box 1567
1820 Montreux, Switzerland
Phone: 011-41-21-966-14-00
Fax: 011-41-21-966-14-01

AUSTRIA

Stryker-Howmedica Osterreich GmbH

Millenium Tower
Handelskai 94-96
120 Wien
Austria
Phone: 011-43-1-240-27-6400
Fax: 011-43-1-240-27-6410

BELGIUM

NV Stryker SA

Ikaros Business Park- Fase III
Ikaroslaan 12
1930 Zaventem, Brussels
Belgium
Phone: 011-32-2-717-9210
Fax: 011-32-2-717-9249

DENMARK

Stryker Denmark

Filial of Stryker AB
Postbox 772
1532 Copenhagen
Denmark
Phone: 011-45-33-93-6099
Fax: 011-45-33-93-2069

ENGLAND

Stryker UK Limited

Stryker House
Hambridge Road
Newbury, Berkshire
RG14 5EG, England
Phone: 011-44-1635-556-500
Phone: 011-44-1635-262-400
Fax: 011-44-1635-580-300

FINLAND

Stryker AB, Finland
PO 80 Makelankatu 2
00501 Helsinki
Finland
Phone: 011-35-89-774-4680
Fax: 011-35-89-774-46820

FRANCE

Stryker France SA
ZAC - Avenue de Satolas Green
69330 Pusignan
France
Phone: 011-33-472-45-36-00
Fax: 011-33-472-45-36-99

GERMANY

Stryker Howmedica GmbH
Dr. Homer Stryker Platz 1
47228 Duisburg
Germany
Phone: 011-49-2065-837-0
Fax: 011-49-2065-837-837

GREECE

Stryker Hellas EPE
455 Messogion ave
153 43 Agia Paraskevi
Athens, Greece
Phone: 011-30-2-10-600-32-22
Fax: 011-30-2-10-600-48-12

ITALY

Stryker Italia Srl
Via Ghisalba 15B
00188 Roma
Italy
Phone: 011-39-06-33-05-41
Fax: 011-39-06-33-614-067

MIDDLE EAST / NORTH AFRICA

Stryker Osteonics SA
Twin Towers
11th Floor, Suite 1101 & 1102
P.O. Box 41446
Baniyas Road
Dubai, Deira, UAE
Phone: 011-97-14-222-2842
Fax: 011-97-14-224-7381

NETHERLANDS

Stryker Nederlands BV
(P.O. Box 13, 4180 BA Waardenburg)
4181 CD Waardenburg
The Netherlands
Phone: 011-31-418-569-700
Fax: 011-31-418-569-777

NORWAY

Stryker Norway
Norsk Filial
Nedre Vollgate 3
0158 Oslo
Norway
Phone: 011-47-22-42-22-44
Fax: 011-47-22-42-22-54

POLAND

Stryker Polska Sp. ZO.O
Kolejowa 5/7
01-217 Warszawa
Poland
Phone: 011-48-22-434-88-50
Fax: 011-48-22-434-88-60

PORTUGAL

Stryker Portugal Produtos Medicos, LTDA.
Avenida Marechal Gomes da Costa, 35
1800-255 Lisboa
Portugal
Phone: 011-35-1-21-839-49-10
Fax: 011-35-1-21-839-49-19

ROMANIA

Stryker Osteonics Romania S.R.L.
19, Leonida Str.
District 2
7000 Bucharest
Romania
Phone: 011-40-2-12-12-11-22
Fax: 011-40-2-12-12-11-33

SOUTH AFRICA

Stryker Osteonics PTY. LTD.
3 Susan Street
Strydom Park
Johannesburg, 2194 - South Africa
Mailing Address:
P.O. Box 48039
2129 Roosevelt Park
Johannesburg, South Africa
Phone: 011-27-11-791-4644
Fax: 011-27-11-791-4696

SPAIN

Stryker Howmedica Iberica S.L.U.
Manual Tovar, 35
28034 Madrid
Spain
Phone: 011-34-91-728-35-00
Phone: 011-34-91-358-20-44
Fax: 011-34-91-358-07-48

SWEDEN

Stryker AB/Scandinavia
Box 50425
SE-204 14 Malmo
Sweden
Phone: 011-46-40-691-81-00
Fax: 011-46-40-691-81-91

European Representative

Stryker EMEA RA/QA Director
Stryker France
ZAC Satolas Green Pusignan
Av. De Satolas Green
69881 MEYZIEU Cedex
France

stryker[®]

3800 E. Centre Ave, Portage, MI 49002
(800) 327-0770
www.stryker.com

CE

JH 11/06 1080-600-171 REV D