

Scale use on Prime Series® Stretchers



Step 1: Zero the Scale

Push and hold the Zero button until the display reads “rel” (release) then release. Wait until the screen stops flashing “000.0”.

The stretcher must be zeroed before a patient can be properly weighed. Also, for the most accurate patient weight, the stretcher must be in high height, cannot exceed a 5° trend, and the surface must be flat with the patient remaining still.



Warning:

Do not use the scale system option or electric lift option or electric litter option adjacent to or tacked with other devices. If adjacent or stacked use is necessary, the scale system option or electric lift option or electric litter option should be observed to make sure that all devices are operating normally in the configuration in which they will be used.


Do not operate the scale system while using the foot supports. Inaccurate readings may occur.

Step 2: Weigh the Patient

Push Weigh button to generate weight of the patient.

Lb/Kg button

Push to toggle between patient weight in pounds or weight in kilograms.

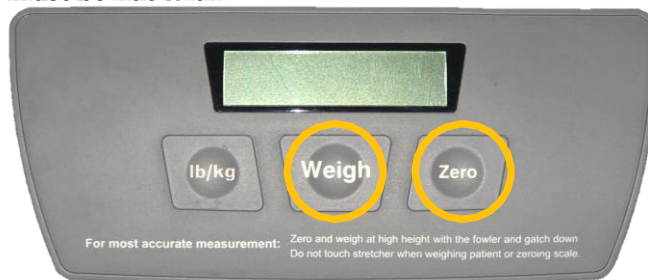

Caution:

Do not use the scale for patients under 50 lb.

Please refer to IFU for scale accuracy.