

Prime Connect™ scale system quick start guide



stryker

Step 1: Zero the scale

Press and hold the Zero button until "Release" displays on the screen.

Note: For the most accurate patient weight, lower the backrest and Gatch so the patient surface is flat. For products with foot end push handles, raise the foot end push handles before you operate the scale system.

Warning:

- Do not use the scale system, electric lift, or electric litter options adjacent to or stacked with other devices. If adjacent or stacked use is necessary, the scale system, electric lift, or electric litter options should be observed to make sure that all devices operate in the configuration in which they will be used.
- Do not operate the scale system while using the foot supports. Inaccurate readings may occur.

Caution:

- Do not use the scale for patients under 50 lb (22.7 kg).
- Do not use the scale system reading as a reference for medical treatment.

Step 2: Weigh the patient

Once the patient is on the stretcher, press the scale button (circled below) to generate the patient's weight.

lb/kg button

The patient's weight displays in kilograms. Press and hold the lb/kg button to view the patient's weight in pounds.



Refer to your Prime Series Stretcher Operations Manual for additional product information:
1105-309-001 (Fifth Wheel)
1115-309-001 (Big Wheel)
1125-509-001 (Zoom Motorized Drive)

Prime Connect™ bed exit alarm

Step 1: Zero the scale

See page 1.

Step 2: Arm bed exit (zone 1 or 2)

Zone 1

- Position the patient on the stretcher.
- Press the arm/disarm button (circled below) to arm bed exit.
- The Zone 1 icon and strip light illuminate green.



or

Zone 2

- Position the patient on the stretcher.
- Press the arm/disarm button twice within three seconds to arm bed exit.
- The Zone 2 icon and strip light illuminate green.



Step 3: Disarm bed exit

Press and hold the arm/disarm button (circled above). The zone icon and strip light will turn off.

Warning:

- Do not use the scale system, electric lift, or electric litter options adjacent to or stacked with other devices. If adjacent or stacked use is necessary, the scale system, electric lift, or electric litter options should be observed to make sure that all devices operate in the configuration in which they will be used.
- Do not operate the bed exit system while using the foot supports. Inaccurate readings may occur.

Caution:

- Do not use the scale for patients under 50 lb (22.7 kg).