INT/R709-WO/6-05/17

# Mistral-Air®

Forced Air Warming

- EN** Instructions for use
- CS** Návod k použití
- DA** Brugsanvisning
- DE** Gebrauchsanleitung
- EL** Οδηγίες χρήσης
- ES** Instrucciones de uso
- FI** Käyttöohjeet
- FR** Instructions d'utilisation
- IT** Istruzioni per l'uso
- JA** 使用説明書
- KO** 사용 지침
- NL** Gebruiksaanwijzing
- NO** Bruksanvisning
- PL** Instrukcja użytkowania
- PT** Instruções de utilização
- RU** Инструкция по эксплуатации
- SR** Uputstvo za upotrebu
- SV** Bruksanvisning
- TR** Kullanım talimatları
- ZH** 使用说明

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## EN Instructions for use

- Unfold and position the blanket with the BLUE side towards your patient
- 1a. If applicable, apply the tape to the dry skin, free of wrinkles and press firmly to increase adhesion
- 1b. Remove the tape from the skin carefully in a movement parallel to the skin
- 1c. Pull the perforated cords and tie to fixate the blanket, if applicable
2. Use the face shield, if applicable
2. Twist the Mistral-Air<sup>®</sup> hose end into the air inlet of the blanket
3. Secure the hose by connecting the hose clamp

**CONTRAINDICATIONS**  
Do not use the device distal to arterial cross clamping  
Do not apply heat to ischemic limbs!

- WARNINGS**
- To be used with a Mistral-Air<sup>®</sup> Warming Unit
  - Do not leave the patient unmonitored during forced air warming
  - Do not use if the Mistral-Air<sup>®</sup> Warming Unit is

alarming or if casing is cracked/damaged  
Warming transdermal medications (patches) can increase drug delivery, resulting in possible harm to the patient  
Do not cover the patient's thorax during cardioversion or defibrillation therapy  
If applicable, only use the face shield on ventilated patients  
Read the Mistral-Air<sup>®</sup> Warming Unit user manual before use

- CAUTIONS**
- Check patient's temperature and skin condition at least every 15 minutes, the hospital protocol is leading
  - Only the blue side should be towards the skin of the patient
  - Adjust the temperature of the Warming Unit when the therapeutic goal is reached, or a vital sign of instability occurs (also notify the physician)
  - Patients' skin must be dry to avoid evaporative cooling

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