



CHG[®] Quick Reference Guide

HOSPITAL BEDS Security Lock-Out

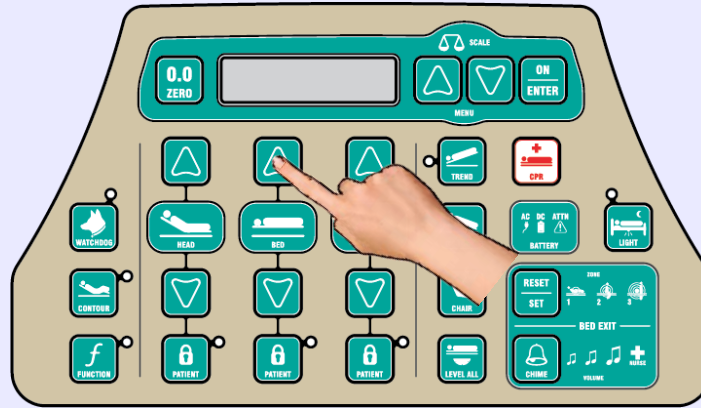

1. Security Lock-Out Feature User Passcode

The User Passcode must first be selected by the Facility. The User Passcode must be between 1 and 6 digits long. The digits are interpreted from a virtual numeric keypad superimposed over the footboard staff control buttons. Image the numbers 1 2 3, 4 5 6, 7 8 9, just as they would appear on a telephone:

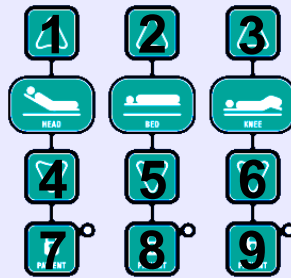

2. Activating/De-Activating the Security Lock-Out Feature

To Enable any/all bed functions, the bed must be unlocked.

To unlock the Security Lock-Out:

1. Enter the 1 to 6 digit User Passcode (Facility Selected Code).
2. Once the User Passcode has been successfully entered, an audible “chirp” will sound and the three Patient Lock-Out buttons will begin to flash to indicate that the bed is now unlocked. All bed functions are now available from all control locations.

IMPORTANT: There is no feedback during this step. If you make a mistake, or if nothing happens after the User Passcode has been entered, simply re-enter the User Passcode starting at the first digit again. There is no need to “cancel” or abort the procedure if you make a mistake while entering the User Passcode.

To re-lock the bed:

When you are finished adjusting the bed positions, press any one of the flashing Patient Lock-Out buttons to re-lock the bed. An audible “chirp” will sound twice to indicate the bed has been re-locked.

OR

The bed will automatically re-lock after 30 seconds of keypad inactivity.

2. Notification of the Security Lock-Out Feature

Once the Security Lock-Out feature has been activated, the LCD will display the following message whenever any button is pressed.



IMPORTANT: ALL bed operations **including the Electronic CPR Function** from any control location (footboard staff control, both siderail controls, and pendant) is restricted.