

Always On Monitoring

The Watchdog feature is used to monitor and indicate if a change has been made to the bed position or the status of most important bed features. On bed power up, the bed defaults to “prepare bed for Watchdog arm” which allows the user 10 seconds (after last key press) to set the desired position of the bed. The Watchdog light will illuminate BLUE.

Position Change

To adjust the position of the bed, simply disarm the Watchdog by pressing-and-holding the WATCHDOG key, then releasing the key when prompted. Release the WATCHDOG key when prompted. The LCD will display a countdown timer of 10 seconds. The timer will reset every time a change is made to the bed. Once all desired bed changes are made, simply allow the countdown timer to elapse. Watchdog Monitoring is now re-established to the new bed position.

Adjusting Monitored Features

By default, the Watchdog system is set to monitor ALL available monitoring points. As patient care situations differ, certain Watchdog monitoring points can be enabled/disabled to allow non-critical bed position changes to happen without triggering a visual warning. The selectable features are:

1. Bed Exit
2. Head Deck Position
3. Knee Deck Position
4. Lock-Outs

To adjust the features monitored by the Watchdog system:

1. Press the ON|ENTER key to access the Menu Options
2. Using the Menu keys, scroll to the Watchdog Menu (Option 3), press ON|ENTER key
3. Using the Menu keys, scroll to the desired feature, press ON|ENTER key
4. Using the Menu keys, set the feature to OFF, press ON|ENTER key

Any change to the selected feature will not trigger a change in the Watchdog Light. To re-enable the feature, simply repeat process and set feature to ON.

Watchdog Light Intensity



To adjust the brightness of the Watchdog light, simply tap the LIGHT key to toggle between the selectable intensities:

- LOW
- MEDIUM
- HIGH
- OFF

Although the Watchdog system is always monitoring, if the Watchdog light brightness is set to OFF, the Watchdog system will not visually display any changes to bed position.

Bed Positon	Watchdog Lights Will Turn Solid YELLOW When...	
Head Angle	Angle changes of approximately ± 5°	
Knee Angle	Angle changes of approximately ± 3°	
Bed Elevation	Height changes of approximately ± 3 inches	
Bed Status	When...	Watchdog Lights Will Be...
Obstruction Sensors	An obstruction is present/detected	Flashing YELLOW
Manual CPR Release	Manual CPR release has been activated	Flashing RED
Patient Lock-Outs	Any change to settings has occurred	Solid YELLOW
Rail Locks	A change to any rail position has occurred	Solid YELLOW

If a change has occurred, to identify what has changed, simply tap the WATCHDOG key. Changes will be displayed on the LCD display.

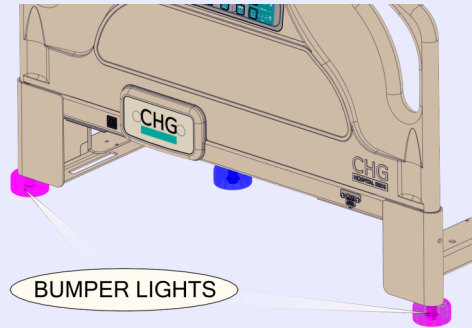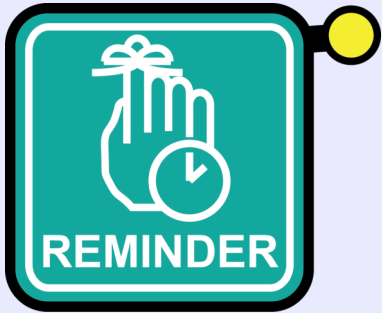

Nursing Reminder™

To provide the best patient care possible, staff may wish to use the Nursing Reminder Mode to identify if a particular patient/bed requires attention (example: patient requires repositioning or a bed needs a change of linens). The bumper lights will illuminate PURPLE when a Nursing Reminder is set.

Reminder Set Points


The Nursing Reminder can be set at 15 minute intervals from 1 minute minimum to 3 hours maximum. Simply tap the ARROW keys until the desired length of time is shown on the LCD display. Press the ON|ENTER key to select the desired time.

Nurse Call Trigger

The Nursing Reminder can be set to trigger a nurse call (if connected) to a central nursing station. After the desired time is selected, the LCD display will ask the user if a nurse call is required. At this point, the user can select either ON or OFF depending on their requirements. Use the MENU keys to select the desired state, then press the ON|ENTER key to confirm.

Nurse Reminder Repeat

The Nursing Reminder will remember the last selected time interval and nurse call selection. To reset the Nurse Reminder, simply tap the NURSE key, and the last selected time interval will be displayed. Once selected, the last nurse call selection will be displayed.

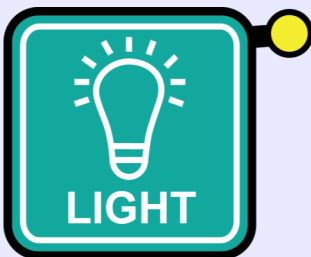

Underbed and Night Lights

The Underbed and Bumper Lights can be used to enhance the room lighting. To control the user selectable light configuration, simply press-and-hold the LIGHT key for 4 seconds. The Lights Menu will be displayed on the LCD display. Use the MENU keys and ON|ENTER key to select various configurations.

SELECTION	SETTINGS						
A. UNDERBED LIGHT	OFF		LOW		MEDIUM		HIGH
B. BUMPER LIGHTS	LT BLUE	LOW	WHITE	LOW	PURPLE	LOW	OFF
		HIGH		HIGH		HIGH	
C. WATCHDOG LIGHTS	OFF		LOW		MEDIUM		HIGH
D. REMINDER LIGHT	LOW				HIGH		
E. BED EXIT LIGHT	ON				OFF		