

EPIC II®
Critical Care Bed
REF 2031

stryker®

Parts Manual



For parts or technical assistance call:
USA: 1-800-327-0770










Table of Contents

English Product Labels.....	4	3002-400-065.....	88
2032-200-003.....	11	2032-130-010.....	89
2032-300-201 / 2032-300-251	18	2032-135-010.....	90
3002-301-201 / 3002-301-251	18	2031-999-140.....	91
3000-200-275.....	19	2032-235-020	92
3001-200-214.....	22	2025-136-021	93
3001-200-340	23	3001-315-012 / 3001-315-016	94
3001-200-345.....	23	3001-315-014 / 3001-315-018	94
3002-201-330	24	3000-300-080.....	95
3002-200-335.....	25	2035-112-000.....	96
2030-200-060.....	26	2035-113-000.....	97
2030-200-065.....	27	2035-112-010 / 2035-113-011	98
5000-002-010	28	0785-035-103	99
3001-200-022	29	2032-140-010.....	100
2031-999-110	30	2035-018-010	101
2031-043-102.....	35	2025-120-010.....	102
2032-300-010	40	2040-111-000	103
2031-999-130.....	51	2025-150-010.....	104
2031-999-131	51	2025-040-010	105
2031-999-034 (Europe).....	52	3003-336-020.....	106
2031-999-035 (British)	52	Recycling Passport	107
2035-432-075.....	53		
3221-300-705	54		
3001-300-775	55		
2031-030-201	56		
2031-030-202	57		
2031-030-204	58		
2031-030-205	59		
2031-400-105 / 2031-400-205	60		
2031-999-161	66		
2032-400-050.....	72		
3001-400-045 / 3001-400-040	73		
3003-402-005.....	74		
3003-402-010.....	75		
3002-400-070.....	76		
3002-400-075.....	77		
3003-400-070.....	78		
3003-400-075.....	79		
3002-400-090.....	80		
2031-400-305 / 2031-400-405.....	81		
2032-401-030	85		
2032-401-025	86		
3002-400-055.....	87		












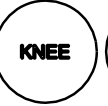





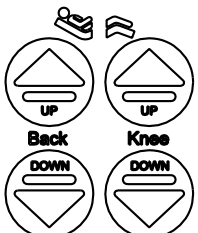





English Product Labels

English

Label Part Number	Label
2031-231-128	    <p>REF Critical Care Bed-Model 2031</p> <p>SN Serial No.</p> <p> Date of Mfg.</p> <p>230V~, 50/60 Hz, 7.0A</p> <p>Max Load = 227 Kg.</p> <p>Rated Duty Cycle: 3 min. On / 30 min. Off</p> <p>This product is in conformity with performance standards for diagnostic x-ray systems and their major components under 21 CFR Part 1020. Patents Applied for.</p> <p> Stryker Medical - Portage, MI 49002-5826, USA</p> <p>Made in U.S.A.</p> <p>CE</p> <p>87VL MEDICAL ELECTRICAL EQUIPMENT UL 60601-1</p>
0988-002-708	<p>CAUTION</p> <p>NO USER SERVICEABLE PART INSIDE. TO REDUCE THE RISK OF ELECTRIC SHOCK, DO NOT REMOVE COVER.</p>  <p>REFER SERVICING TO QUALIFIED SERVICE PERSONNEL.</p>
1550-090-001	<p>CAUTION</p> <p>ADEQUATE GROUNDING RELIABILITY CAN ONLY BE ACHIEVED WHEN THE EQUIPMENT IS CONNECTED TO AN EQUIVALENT RECEPTACLE MARKED "HOSPITAL GRADE"</p>
2011-001-104	<p>DANGER</p> <p>EXPLOSION HAZARD. DO NOT USE IN THE PRESENCE OF FLAMMABLE ANESTHETICS.</p> <p>DANGER</p> <p>RISQUE D'EXPLOSION NE PAS EMPLOYER EN PRESENCE D'ANESTHESIQUES INFLAMMABLES.</p> <p>EXPLOSIONSGEFAHR!</p> <p>NICHT IN DER NAEHE VON FEUERGERAEHRlichen ANASTHESIEMITTENLN VERWENDEN.</p>

English Product Labels






English

Label Part Number	Label
2030-000-151	<div> <div> SIDERAIL CONTROL LIGHTS   </div> <div> BED MOTION LOCK   </div> <div> SIDERAIL CONTROL LOCKOUTS          </div> <div> BED MOTION     </div> </div> <div> <input type="radio"/> POWER <input type="radio"/> BED MOTION LOCKED <input type="radio"/> BRAKE NOT SET </div>
2035-000-153	<div>   </div>
2035-000-155	<div> PATIENT POSITIONING CONTROLS   </div>
2040-090-100	<div>   500 LB • 227 KG </div>
3000-200-601	<div> BRAKE PRESS TO SET OR RELEASE </div>

[Return To Table of Contents](#)






English Product Labels

English

Label Part Number	Label
3000-300-641	 <p>CAUTION</p> <p>BED MUST BE PLUGGED INTO GROUNDED OUTLET PRIOR TO INITIAL INSTALLATION OF FOOTBOARD.</p>
3000-400-556	<div> <p>WARNING- POWERED BED MECHANISMS CAN CAUSE SERIOUS INJURY. OPERATE BED ONLY WITH PERSONS CLEAR OF MECHANISMS.</p> <p>CAUTION- UNPLUG BED DURING SERVICE OR CLEANING. REFER TO SERVICE AND OPERATIONS MANUALS FOR ADDITIONAL PRECAUTIONS.</p> </div>
3000-500-025	 <p>FOR HOSPITAL STAFF USE ONLY</p>
3000-500-029	<div> <p>CAUTION- POSSIBLE FIRE HAZARD WHEN USED WITH OXYGEN ADMINISTERING EQUIPMENT OF OTHER THAN THE NASAL OR MASK TYPE. LOCK CONTROL AT FOOT OF BED WHEN USING OXYGEN ADMINISTERING EQUIPMENT.</p> </div>
3000-526-003	 <p>PULL OUT FOR CPR</p>
3001-300-603	 <p>CPR RELEASE SQUEEZE HERE</p>
3003-503-901	 <p>⚠ PUSH TO RELEASE SIDERAIL</p>

English Product Labels

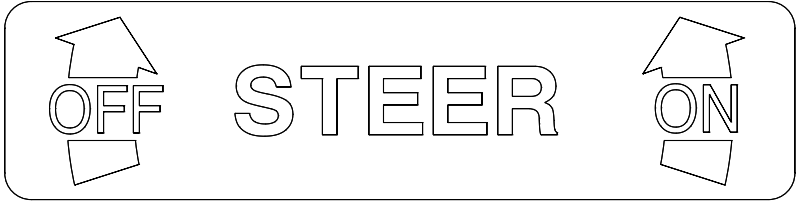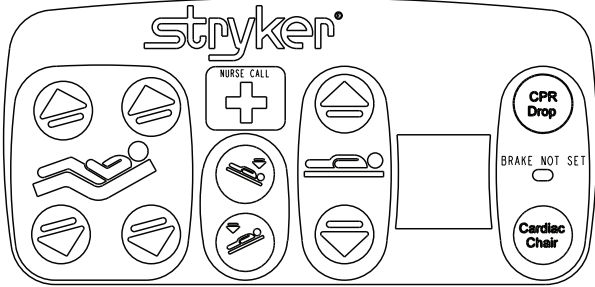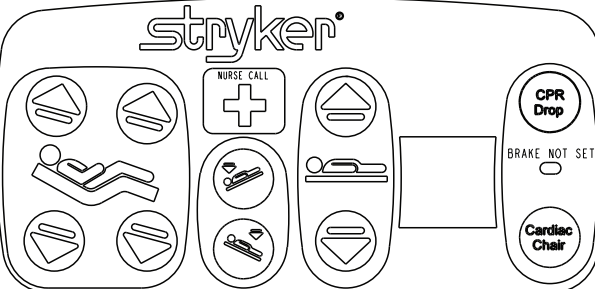
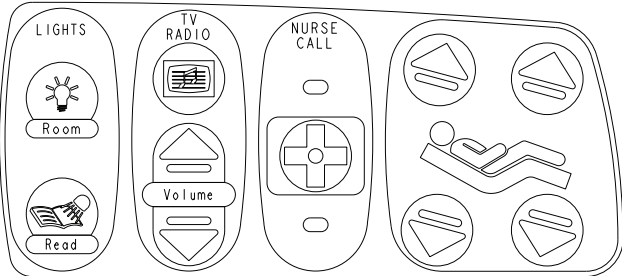
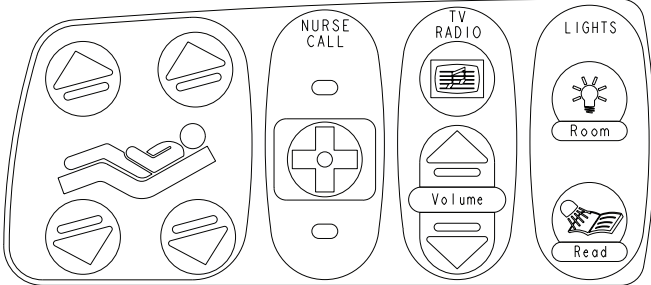


English

Label Part Number	Label
5000-090-013	
2030-000-301	
2030-000-401	
2035-000-102	
2035-000-202	

[Return To Table of Contents](#)


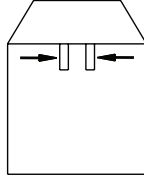


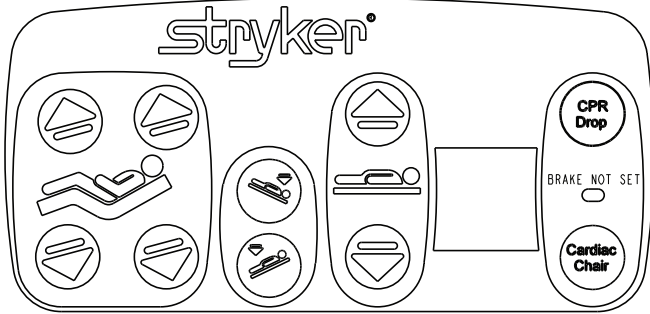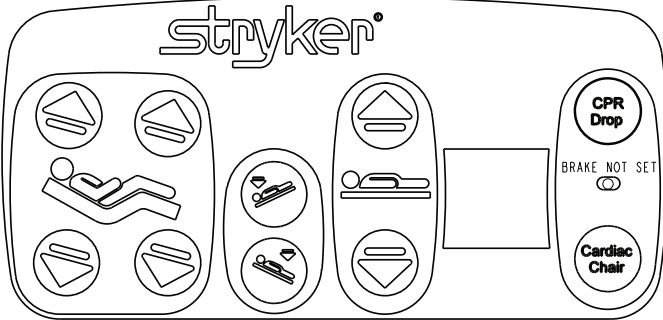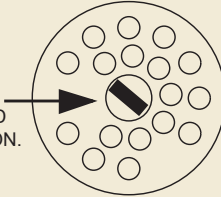English Product Labels

English

Label Part Number	Label
2030-000-300	
2030-000-400	
5000-090-028	<p>NOTE: TO PREVENT A LOW BATTERY CONDITION: WHEN BED IS NOT PLUGGED IN, POSITION THE CORD OUT SWITCH TO THE OFF POSITION.</p> 
2035-112-110	<div> <div>  <p>2035-112-010 Two Stage Permanent I.V. Pole</p> <p>Portage, MI 49002 Made In U.S.A.</p> <p>CAUTION: Not Intended For Use As A Pushing Device</p> </div> </div>
2035-113-111	<div> <div>  <p>2035-113-011 Two Stage Permanent I.V. Pole</p> <p>Portage, MI 49002 Made In U.S.A.</p> <p>CAUTION: Not Intended For Use As A Pushing Device</p> </div> </div>

English Product Labels


English


Label Part Number	Label
3000-300-089	 <p>3000-300-080 I.V. POLE</p> <p>Portage, MI 49002 Made In U.S.A.</p>
1010-023-019	 <p>To Remove Tray, Grasp Latch Pins as Shown</p>
2035-140-025	 <p>2035-140-010 Fowler X-Ray Cassette Holder</p> <p>Portage, MI 49002 Made In U.S.A.</p>
2025-150-002	 <p>2025-150 02 Bottle Holder</p> <p>Portage, MI 49002 Made in the U.S.A.</p>
1010-050-019	<p>NOT INTENDED FOR USE AS A PUSH/PULL DEVICE</p>
1010-050-057	<p>Maximum Weight 40 lbs.</p>


[Return To Table of Contents](#)


English Product Labels


English


Label Part Number	Label
2025-120-005	EQUIPMENT MUST BE STRAPPED TO TRAY
2025-120-006	 <p>The label for the Stryker Defibrillator Tray features the Stryker logo at the top, followed by the product name 'Defibrillator Tray'. At the bottom, it specifies 'Portage, MI 49002' and 'Made In U.S.A.'.</p>
2030-140-002	<p>Warning: Do not use as a Push/Pull device. Maximum Load 45 lbs.</p> <p>Note: Items added and subtracted from the hanger after arming the BED EXIT or SCALE system will affect the sensitivity and accuracy of these systems.</p>

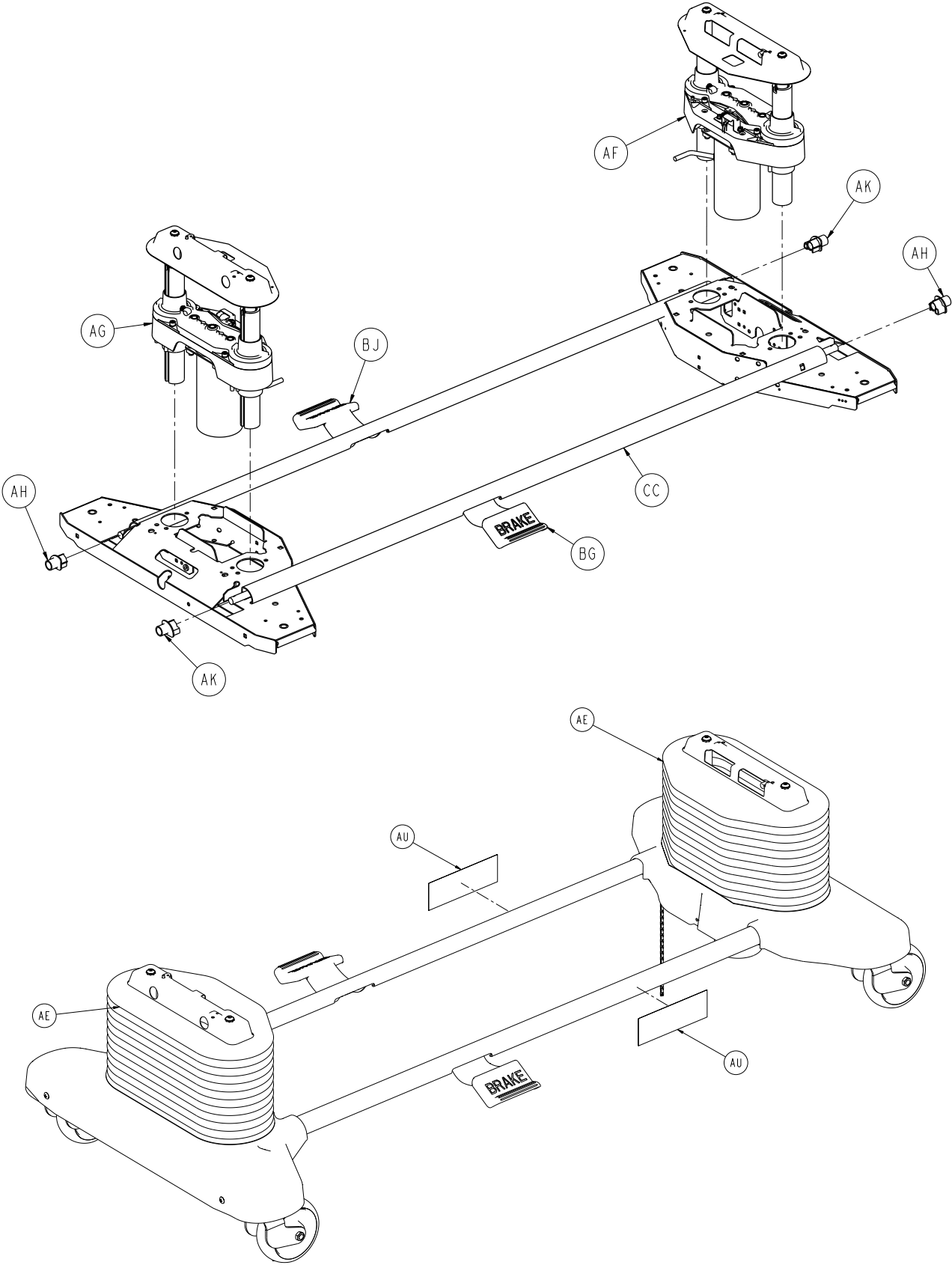


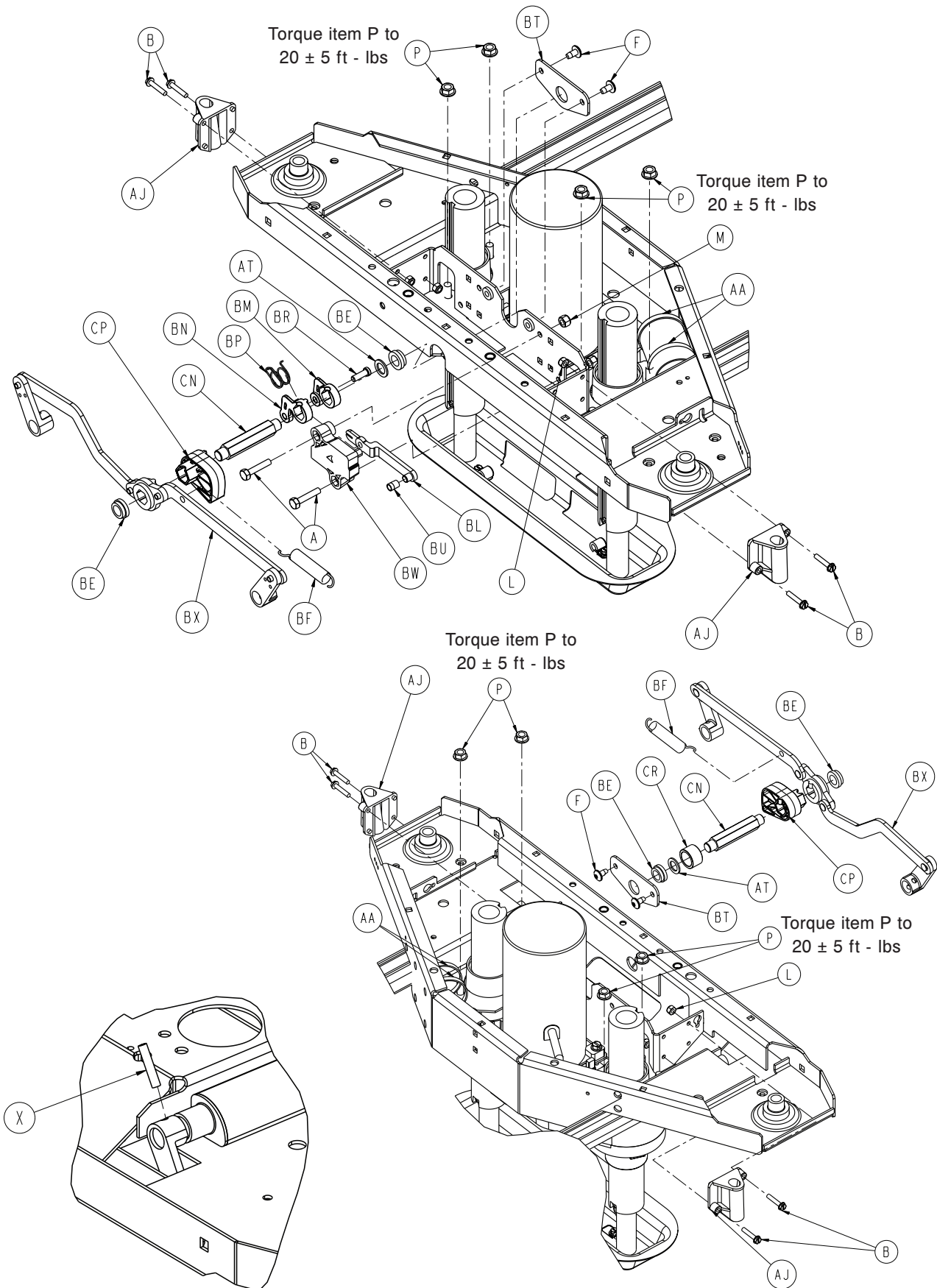

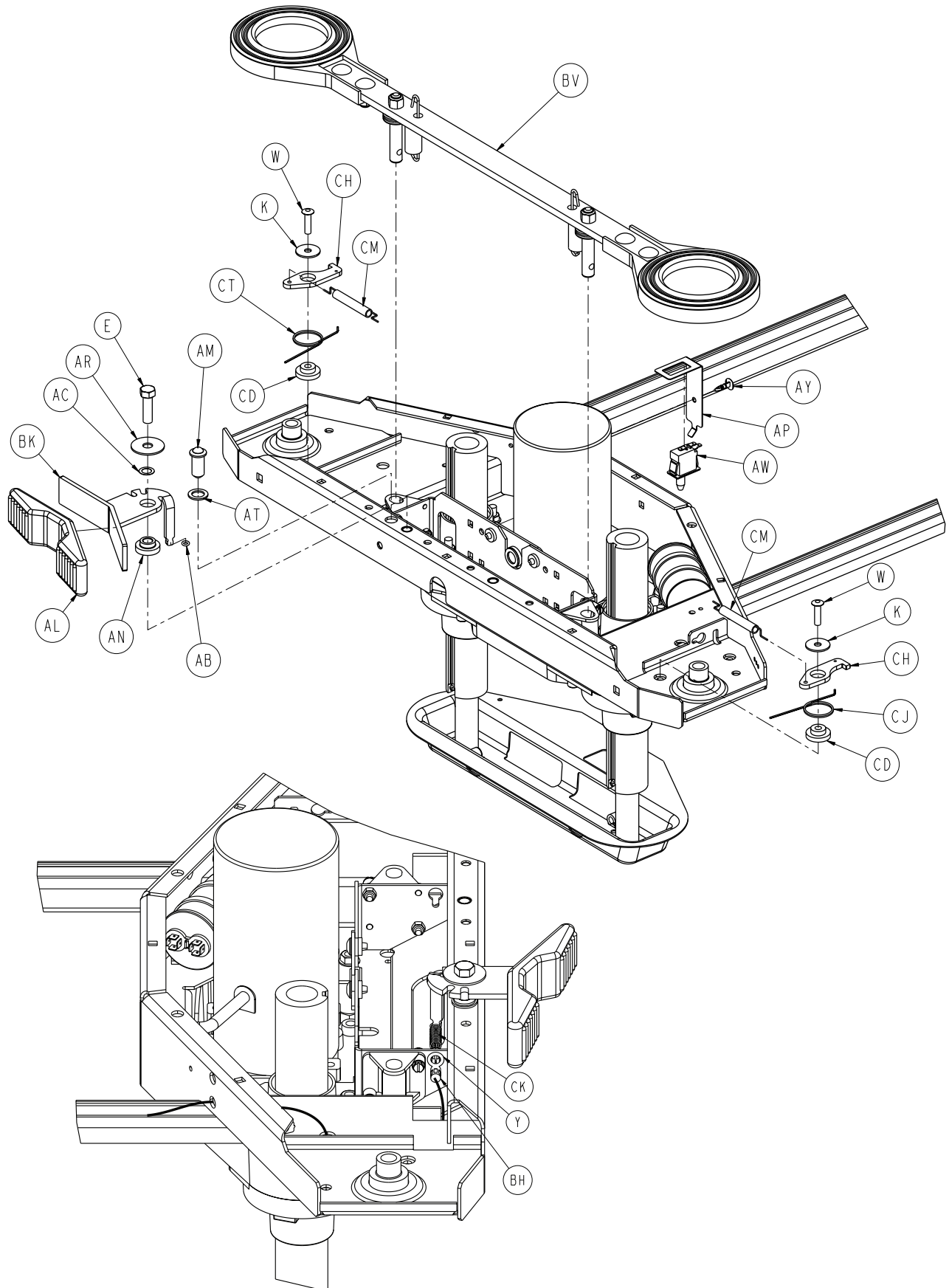

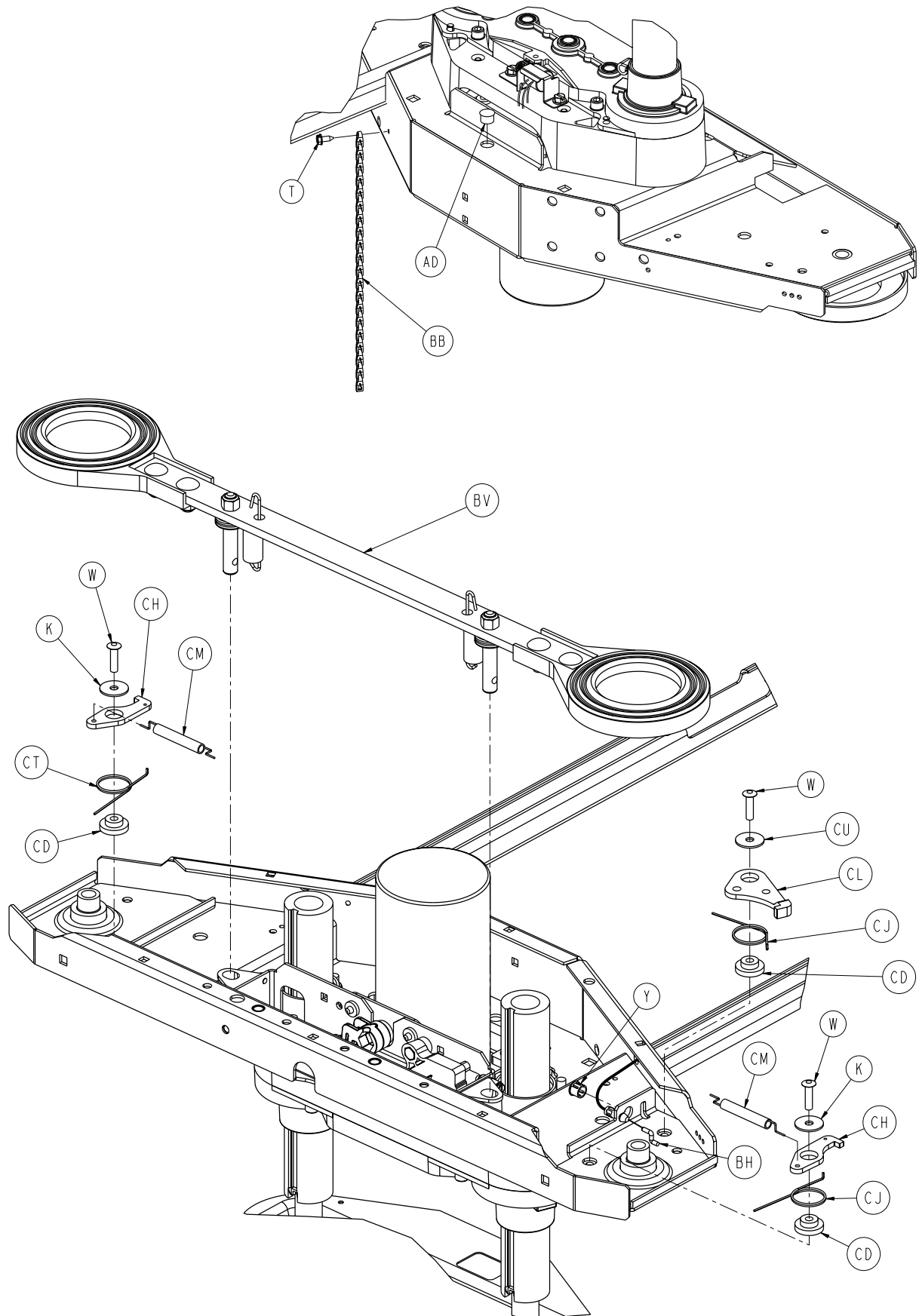

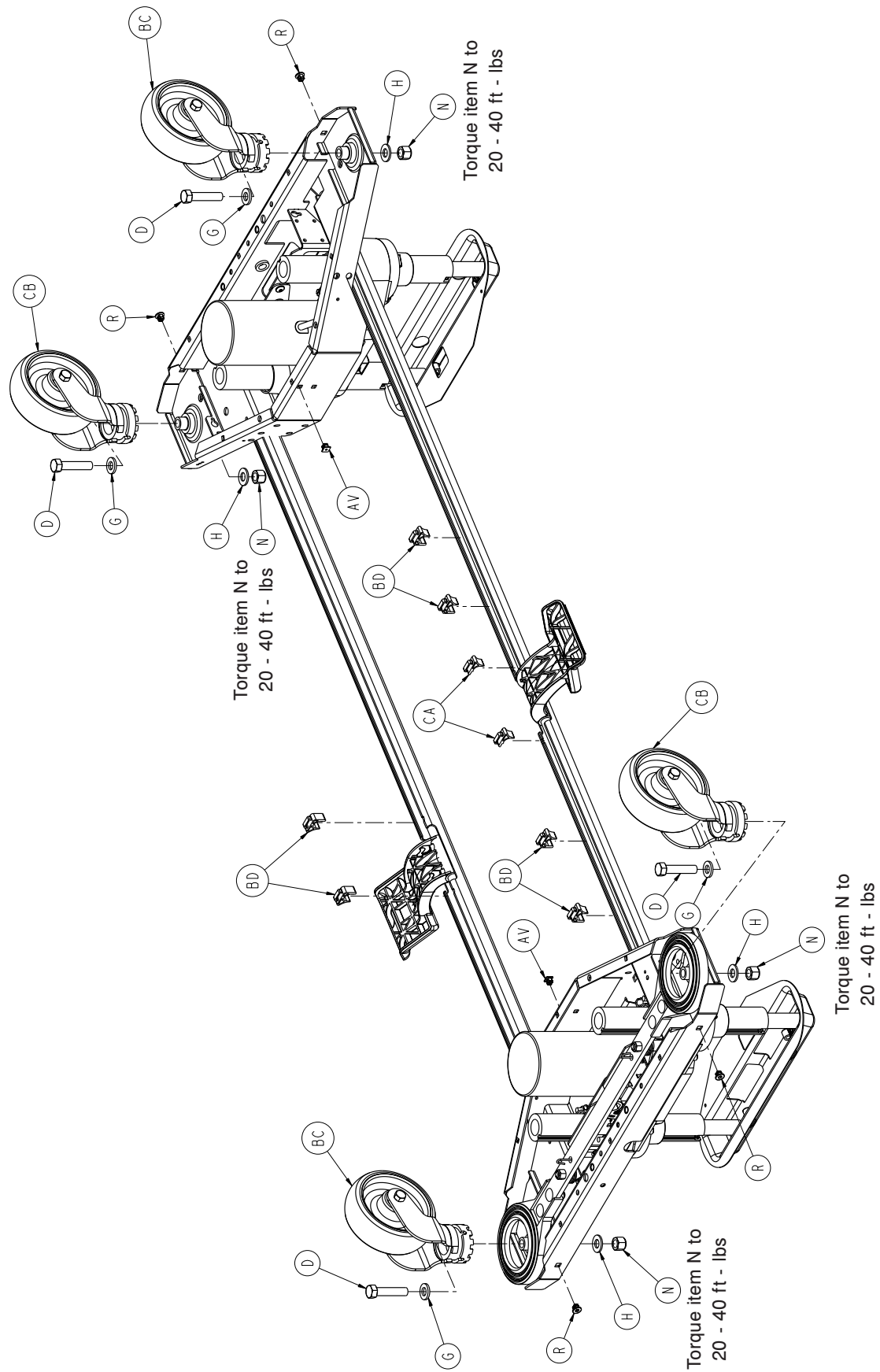


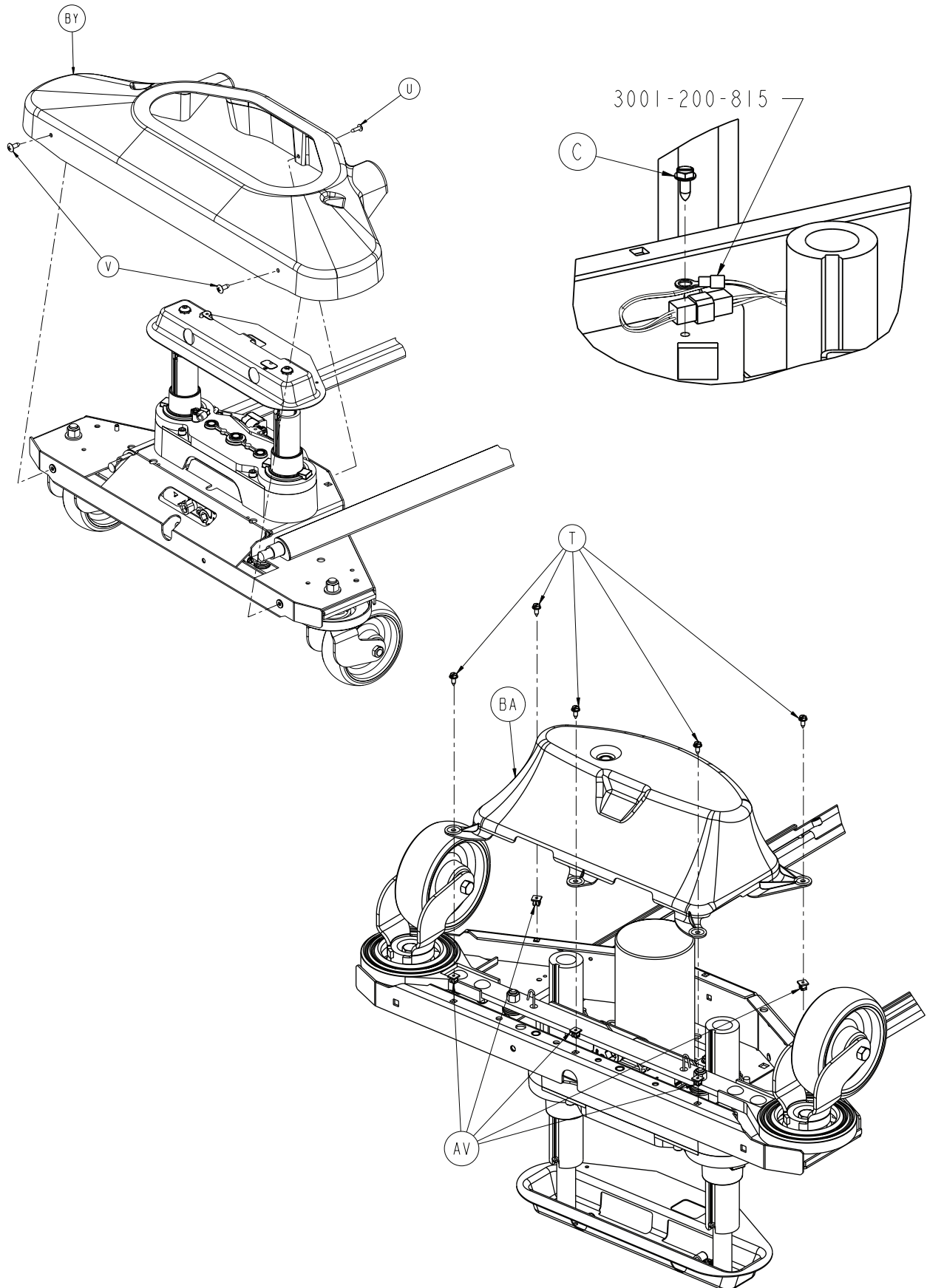

2032-200-003

A	0003-074-000	2	BN	3002-200-306	1
B	0003-122-000	8	BP	3002-200-307	1
C	0003-224-000	2	BR	3002-200-308	1
D	0003-333-000	4	BT	3002-200-314	2
E	0003-349-000	1	BU	3002-200-316	1
F	0007-052-000	4	BV	3006-200-335	2
G	0011-310-000	4	BW	3002-201-301	1
H	0011-333-000	4	BX	3006-200-330	2
K	0011-611-000	4	BY	3002-300-009	2
L	0016-002-000	8	CA	3002-316-000	2
M	0016-028-000	2	CB	2030-200-065	2
N	0016-049-000	4	CC	3006-200-105	1
P	0016-098-000	8	CD	3006-200-339	5
R	0018-036-000	4	CH	3006-200-374	4
T	0023-025-000	12	CJ	3006-200-378	3
U	0023-080-000	2	CK	3006-200-380	1
V	0023-092-000	4	CL	3006-200-381	1
W	0025-191-000	5	CM	3006-200-376	4
X	0026-014-000	4	CN	3002-200-311	2
Y	0030-052-000	2	CP	3006-200-312	2
AA	0038-151-000	4	CR	3002-200-313	1
AB	0045-008-000	1	CT	3006-200-379	2
AC	0052-305-000	1	CU	0011-611-001	1
AD	0059-100-000	2			
AE	2030-001-103	2			
AF	2032-300-201	1			
AG	2032-300-251	1			
AH	3000-200-305	2			
AJ	3000-200-328	4			
AK	3000-200-331	2			
AL	3000-200-336	1			
AM	3000-200-337	1			
AN	3000-200-341	1			
AP	3000-200-343	1			
AR	3000-200-348	1			
AT	3000-200-349	3			
AU	3000-200-602	2			
AV	3000-300-002	12			
AW	3000-300-058	1			
AX	3000-300-113	6			
AY	3000-300-115	1			
BA	3001-200-022	2			
BB	3001-200-052	2			
BC	2030-200-060	2			
BD	3001-200-306	6			
BE	3001-200-317	4			
BF	3001-200-334	2			
BG	3001-200-340	1			
BH	3001-200-342	1			
BJ	3001-200-345	1			
BK	3001-200-371	1			
BL	3002-200-302	1			
BM	3002-200-305	1			


2032-300-201 / 2032-300-251

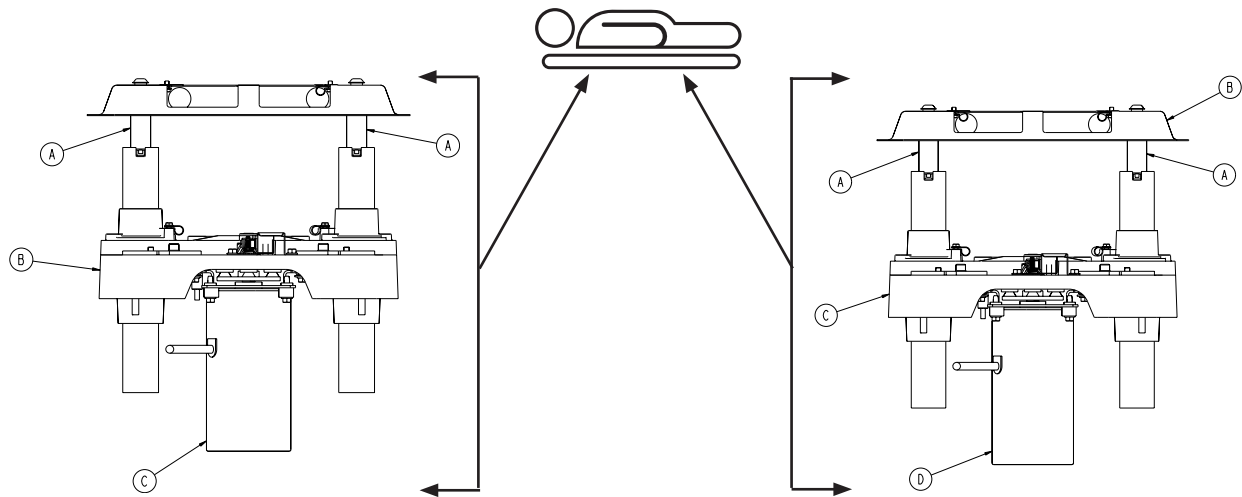


A	2040-200-249	2
B	3002-301-201	1
C	3221-016-603	1

A	2040-200-249	2
B	3002-200-052	1
C	3002-301-251	1
D	3221-016-603	1


3002-301-201 / 3002-301-251

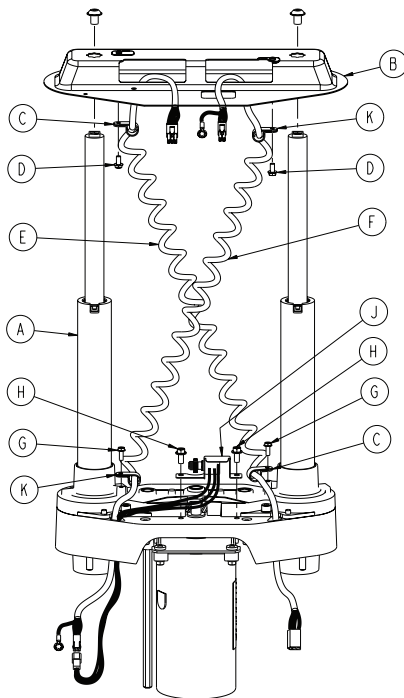


A	3000-200-275	1
B	3002-200-052	1
C	0034-022-000	2
D	0023-164-000	2
E	3001-200-864	1
F	3001-200-815	1
G	0003-128-000	2
H	0003-121-000	2
J	3001-200-240	1
K	0034-381-000	2

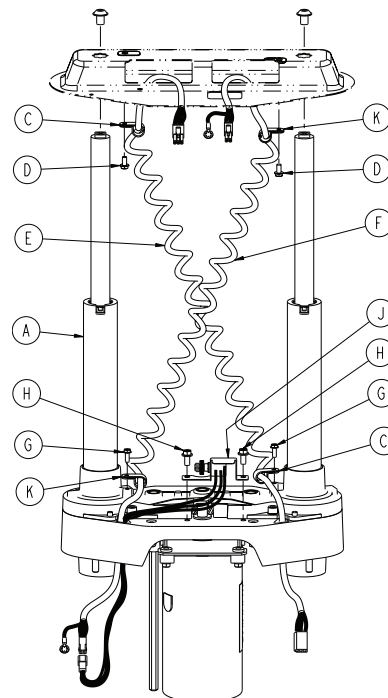


A	3000-200-275	1
C	0034-022-000	2
D	0023-164-000	2
E	3001-200-864	1
F	3001-200-815	1
G	0003-128-000	2
H	0003-121-000	2
J	3001-200-230	1
K	0034-381-000	2

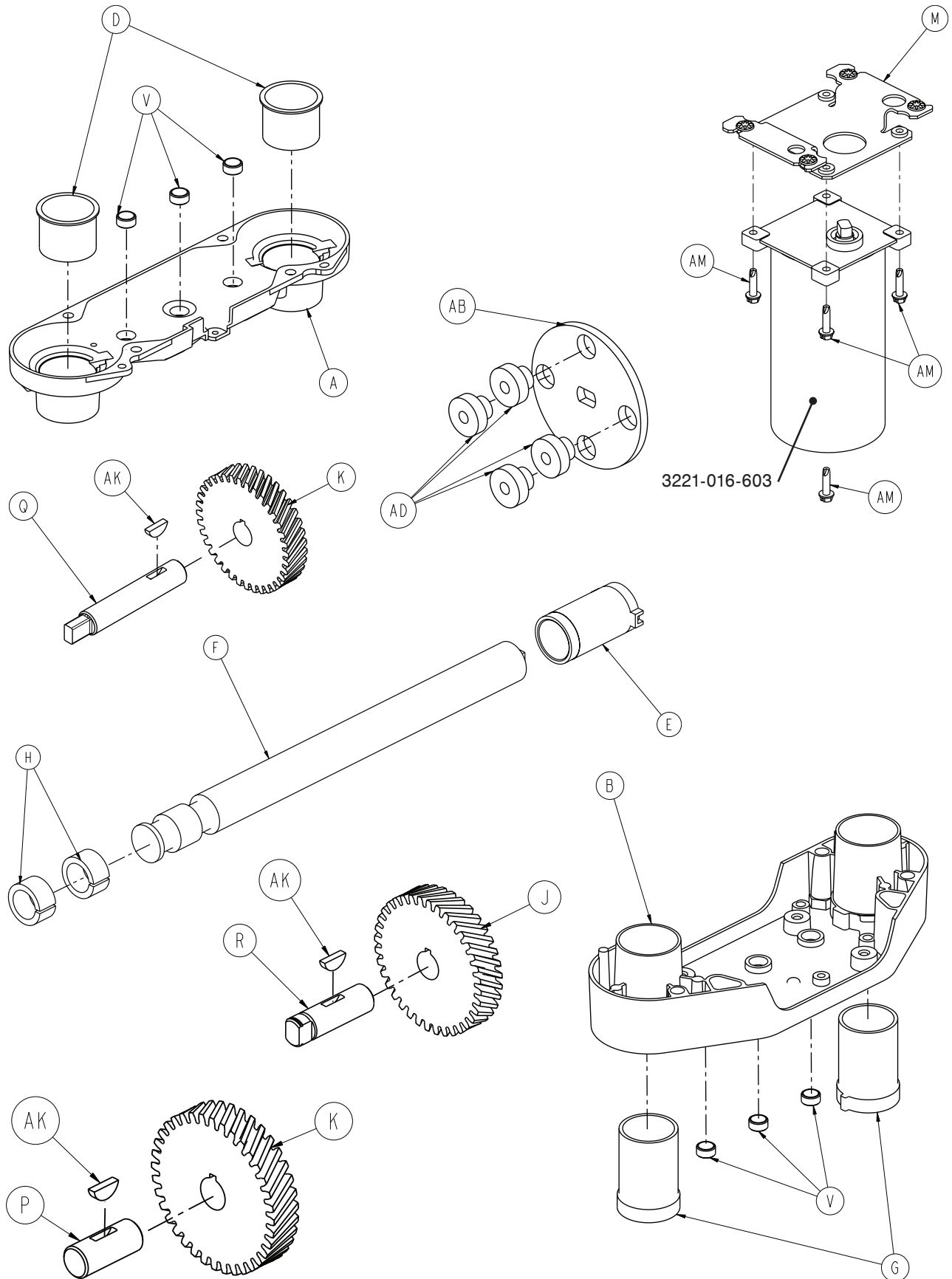

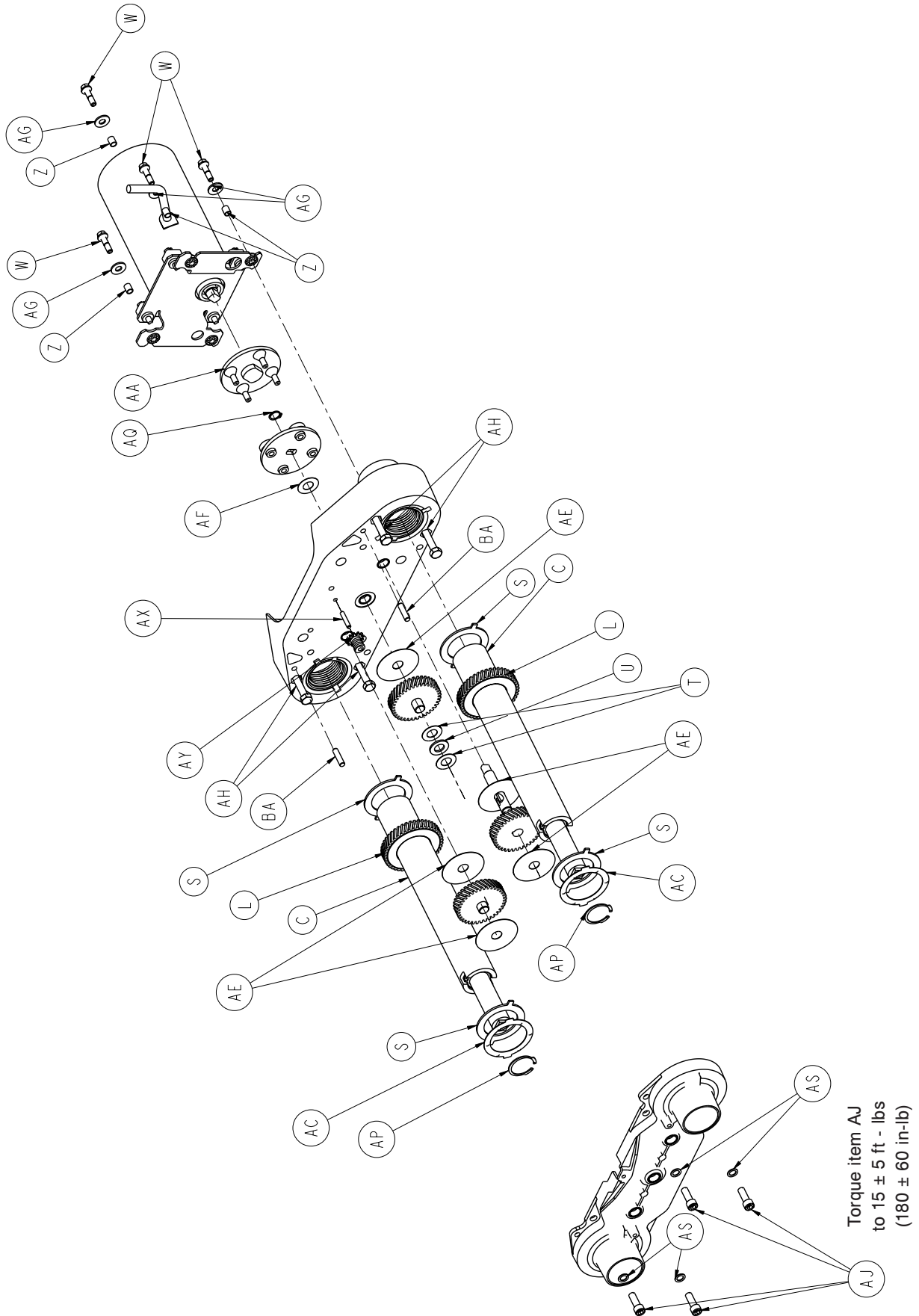


3000-200-275

A	3000-200-201	1
B	3000-200-202	1
C	3000-200-251	2
D	3000-200-204	2
E	3000-200-205	2
G	3000-200-207	2
H	3000-200-208	4
J	3000-200-209	1
K	3000-200-210	2
L	3000-200-252	2
M	3001-200-214	1
P	3000-200-218	1
Q	3002-200-235	1
R	3000-200-220	1
S	3000-200-223	4
T	3000-200-224	2
U	0081-212-000	1
V	3000-200-226	6
W	3001-200-228	4
Z	3001-300-019	4
AA	3000-200-233	1
AB	3000-200-234	1
AC	3000-200-241	2
AD	3000-300-455	4
AE	3000-200-245	5
AF	3000-200-246	1
AG	0011-408-000	4
AH	0003-082-000	4
AJ	0004-213-000	4
AK	0058-044-000	3
AM	0003-331-000	4
AP	0028-121-000	2
AQ	0028-097-000	1
AS	0011-308-000	4
AX	3000-200-239	1
AY	3000-200-216	1
AZ	0051-022-000	4
BA	0026-231-000	2

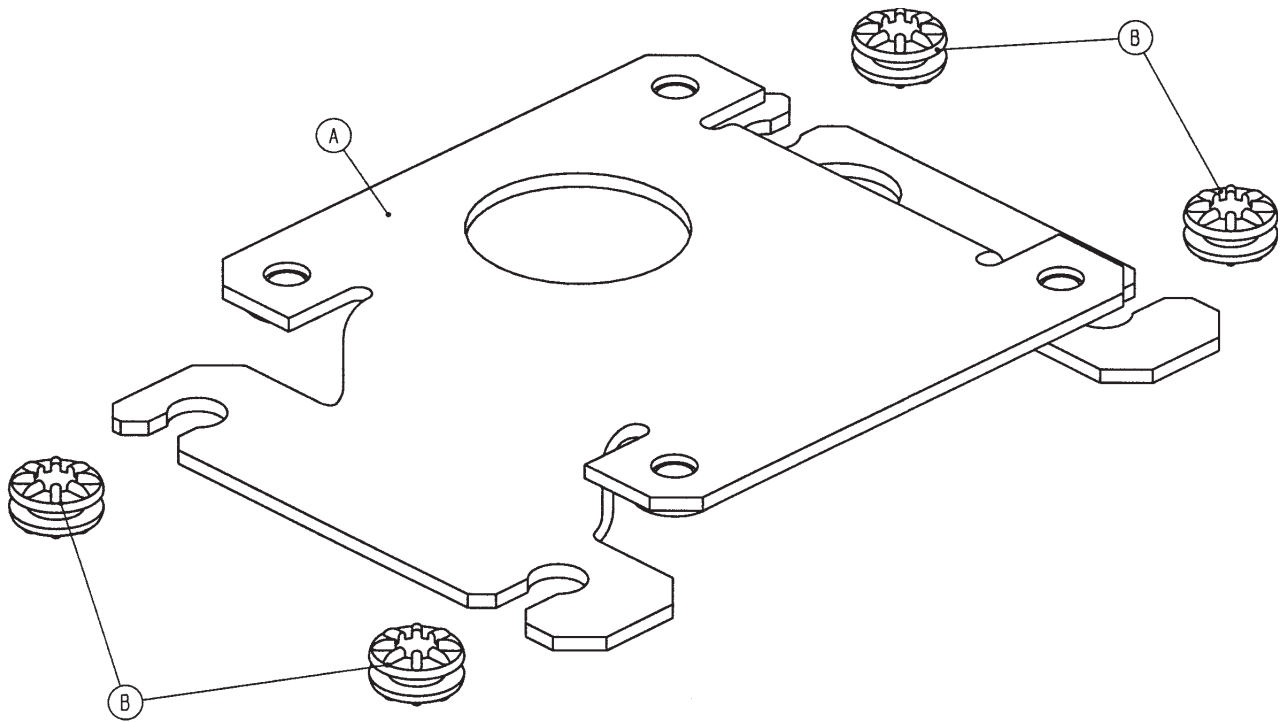


A	3001-200-213	1
B	3000-300-442	4


3001-200-340

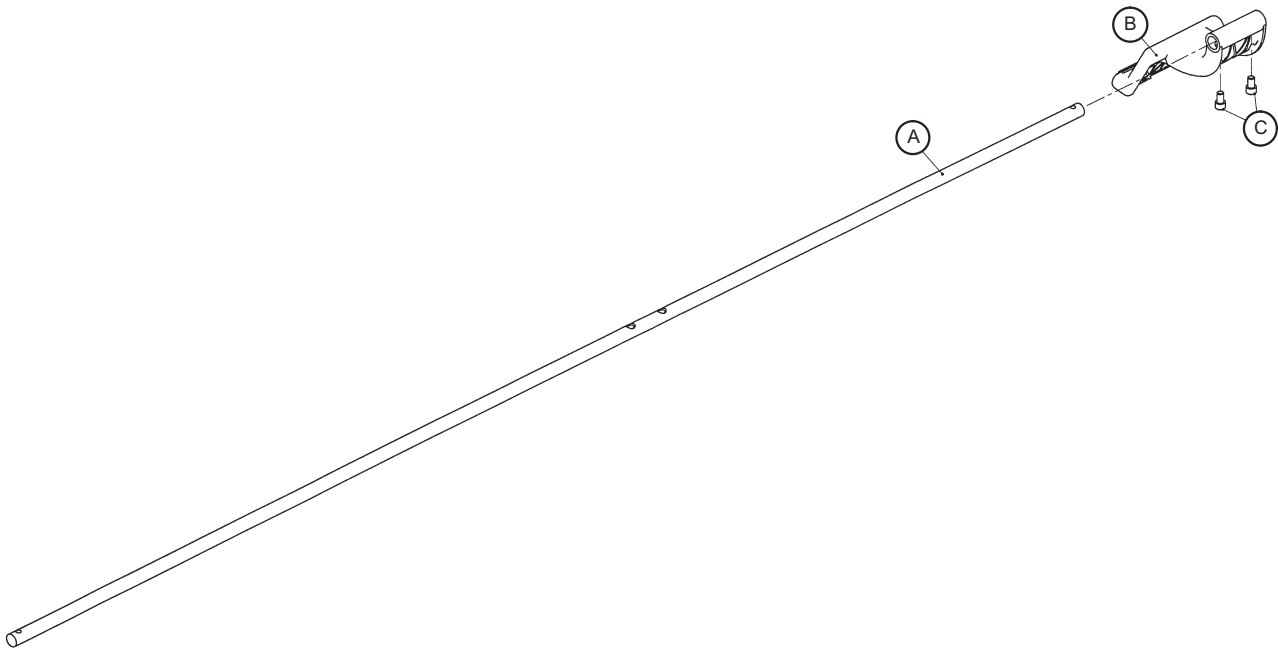


3001-200-345

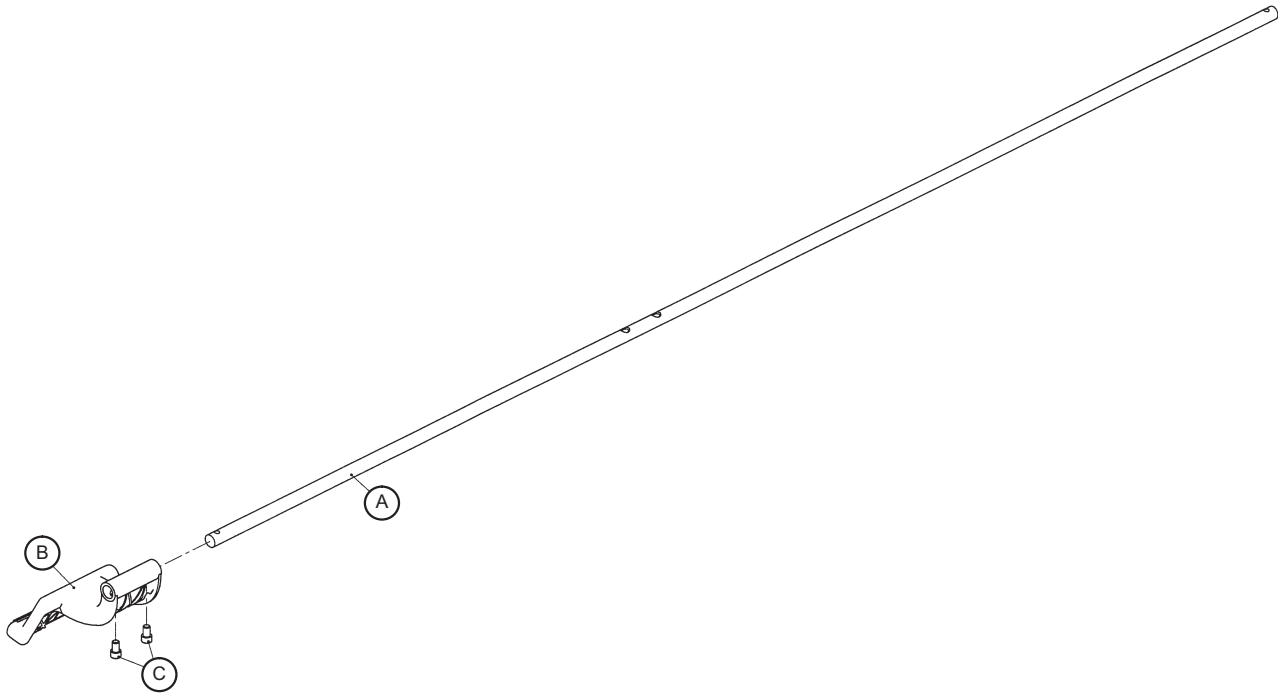


A	3000-200-314	1
B	3001-200-325	1
C	0004-270-000	2

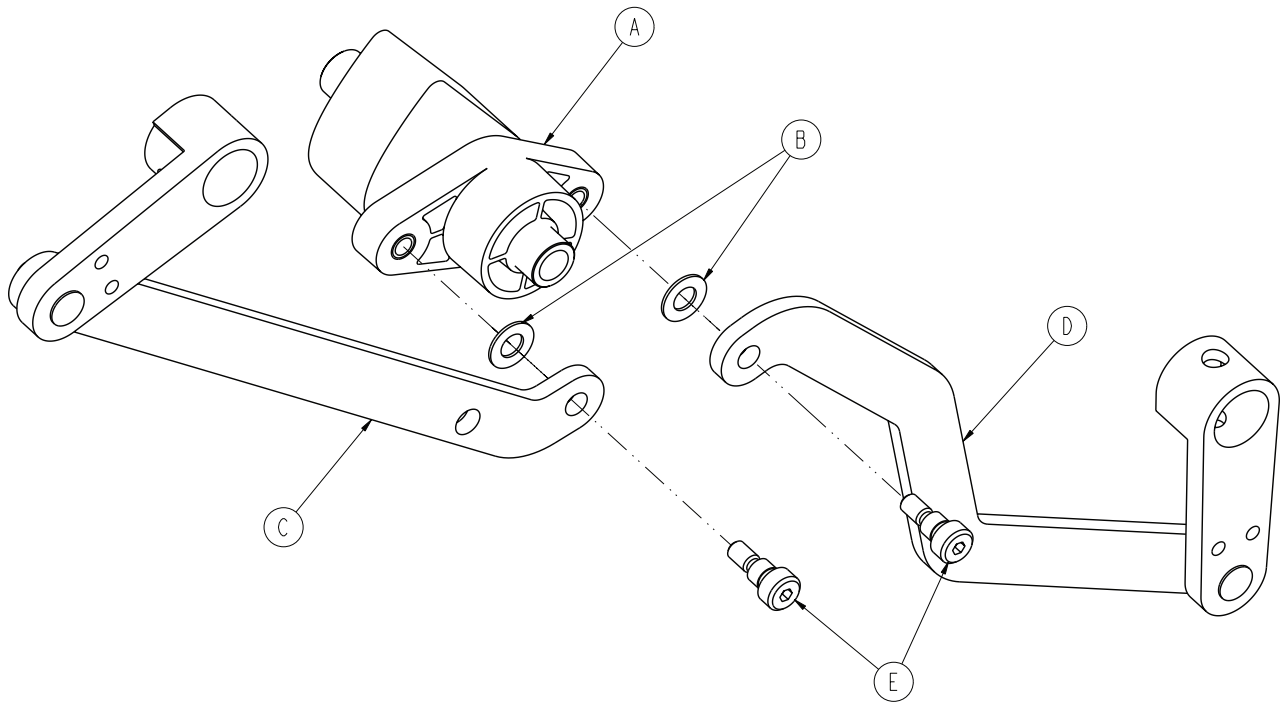


A	3002-201-309	1
B	0014-004-000	2
C	3002-200-331	1
D	3002-200-332	1
E	0002-108-000	2

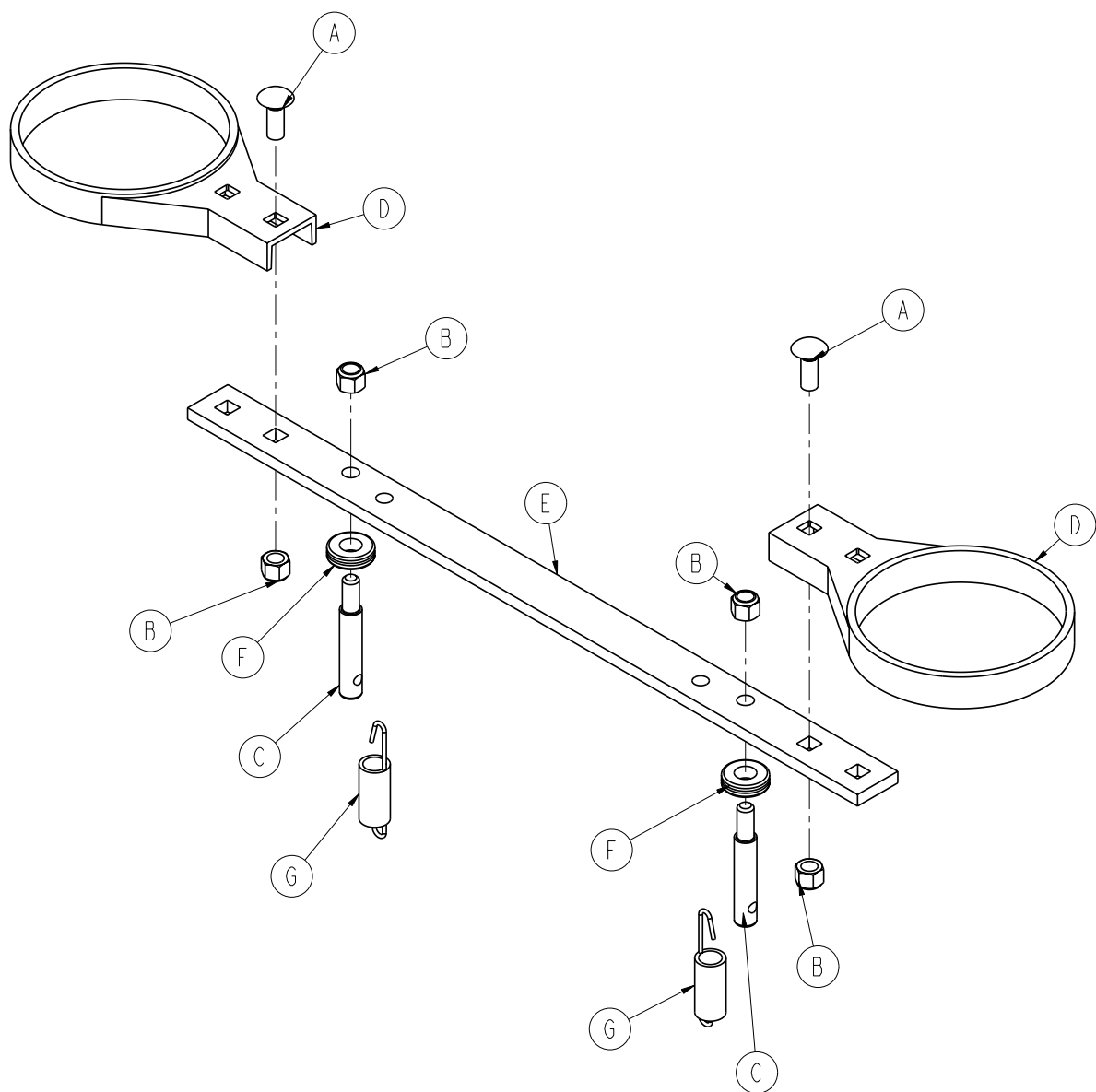


A	0005-018-000	2
B	0016-035-000	4
C	3000-200-318	2
D	3000-200-321	2
E	3000-200-323	1
F	3000-200-324	2
G	3002-200-310	2

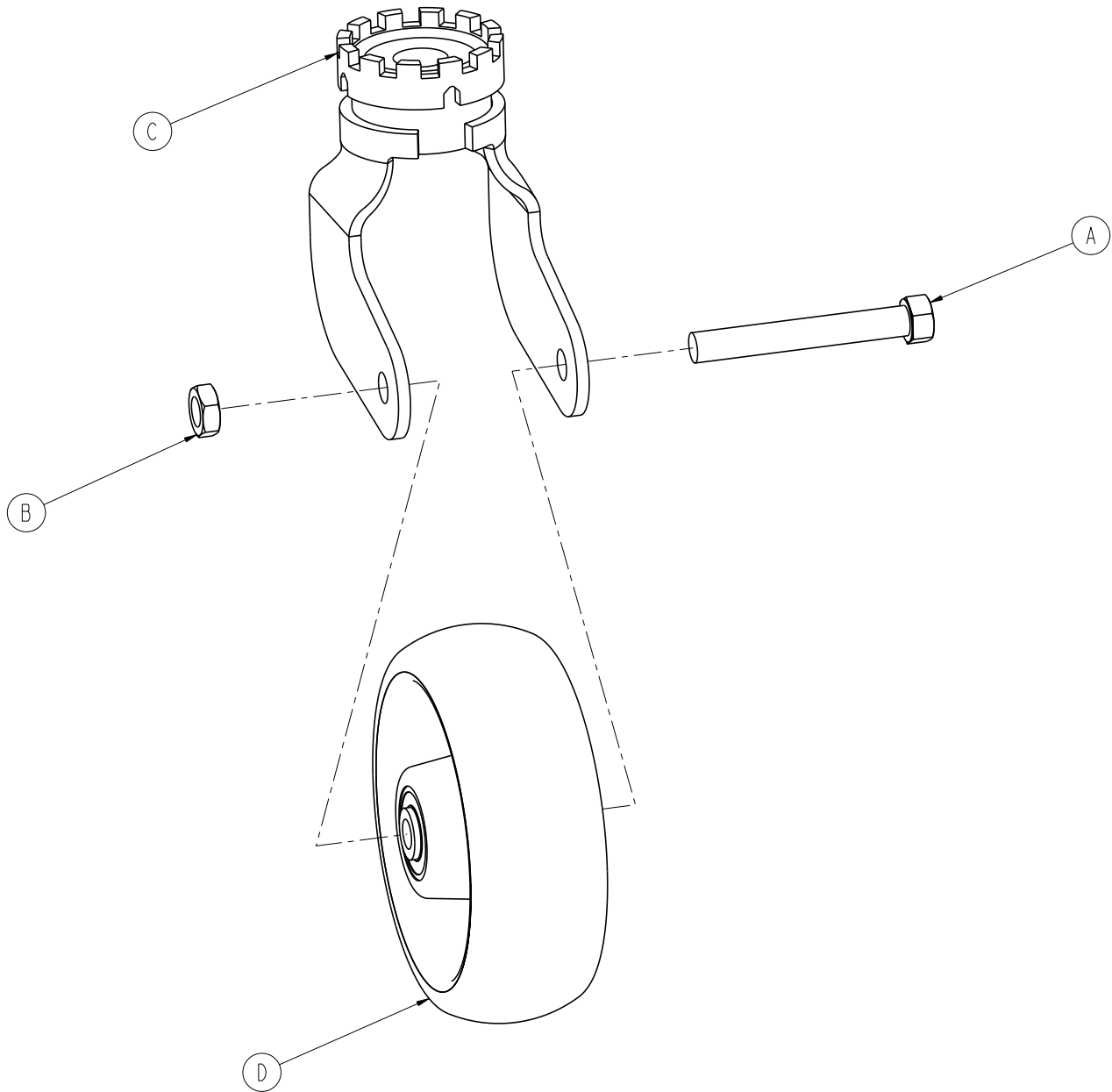


A	0003-342-000	1
B	0016-060-000	1
C	2030-200-059	1
D	5000-002-010	1

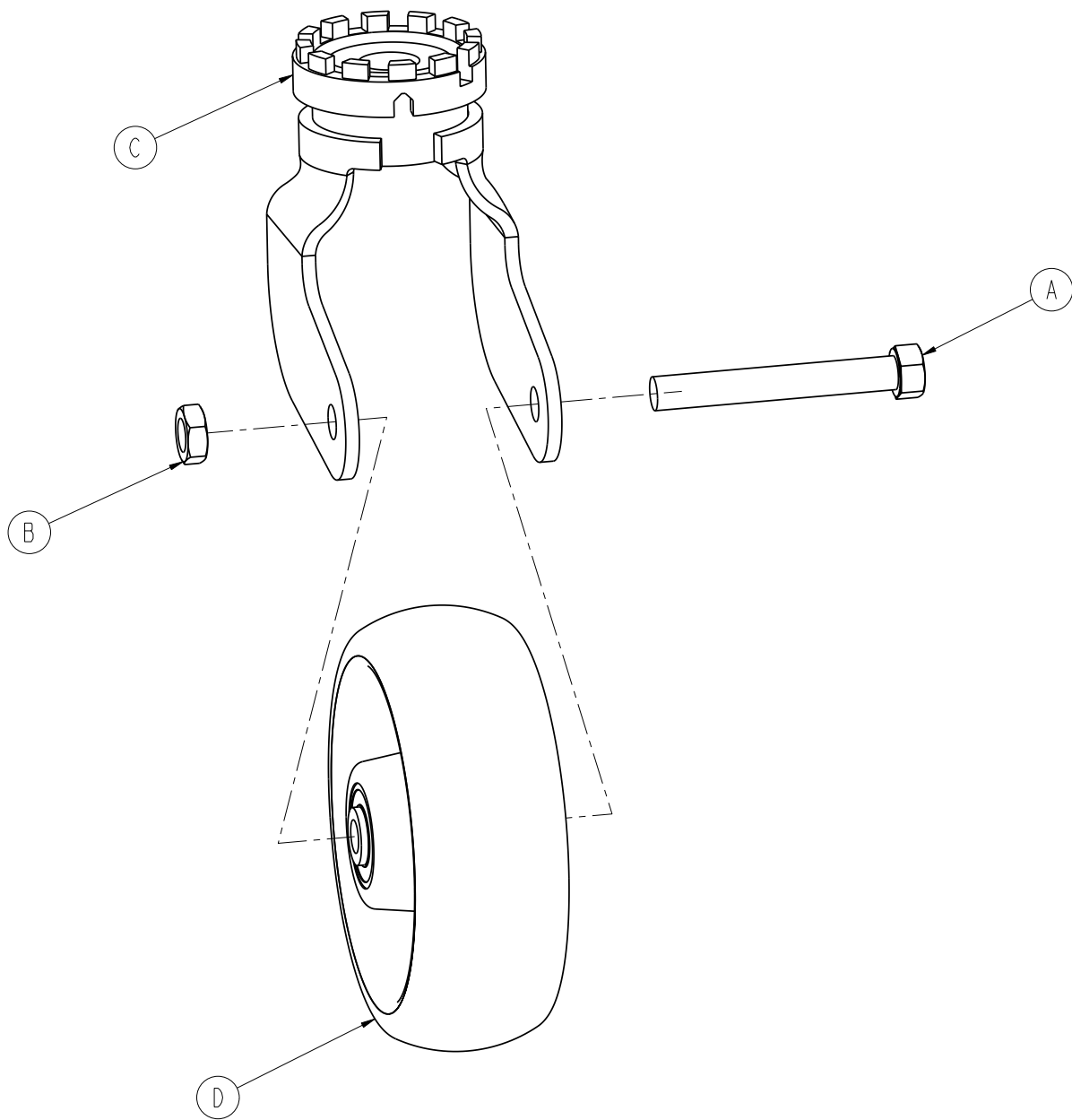


A	0003-342-000	1
B	0016-060-000	1
C	2030-200-063	1
D	5000-002-010	1


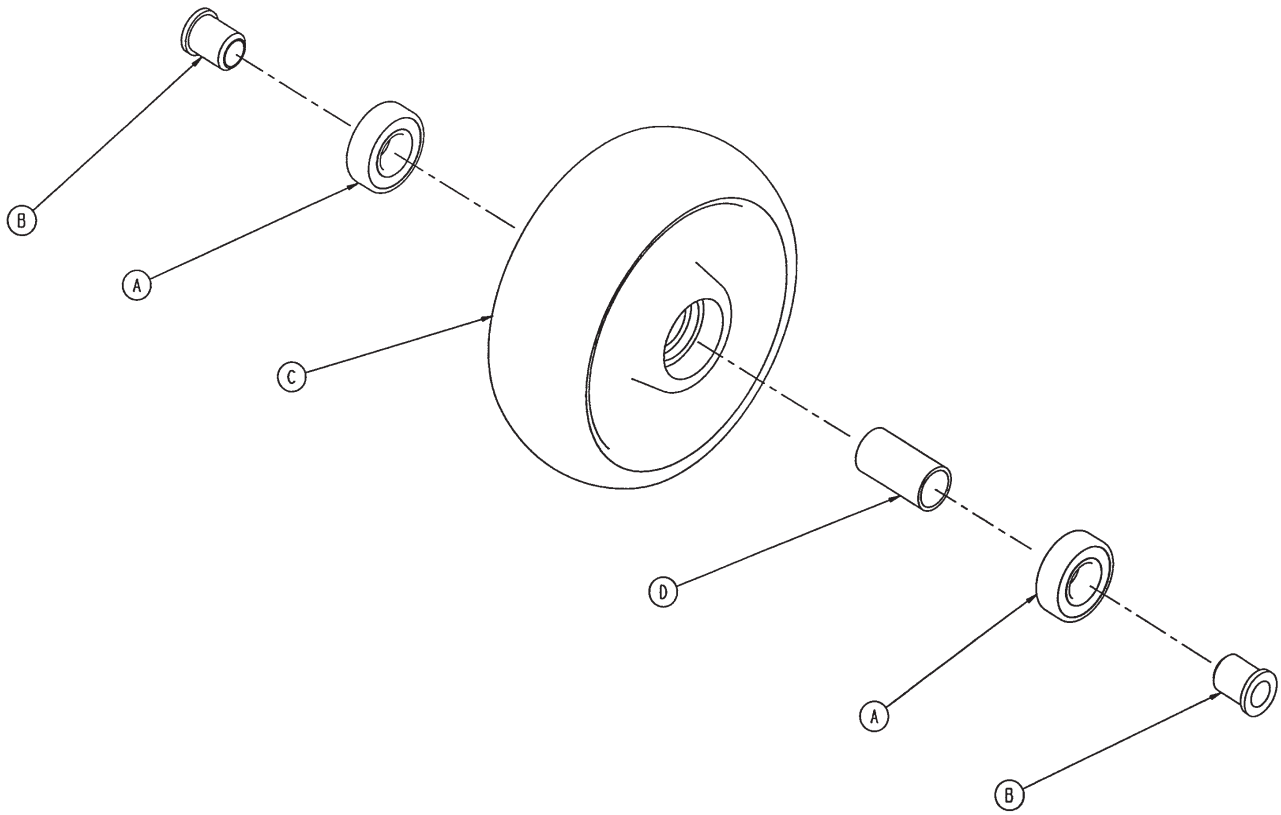



A	0081-226-000	2
B	0715-001-255	2
C	5000-002-020	1
D	6060-002-046	1

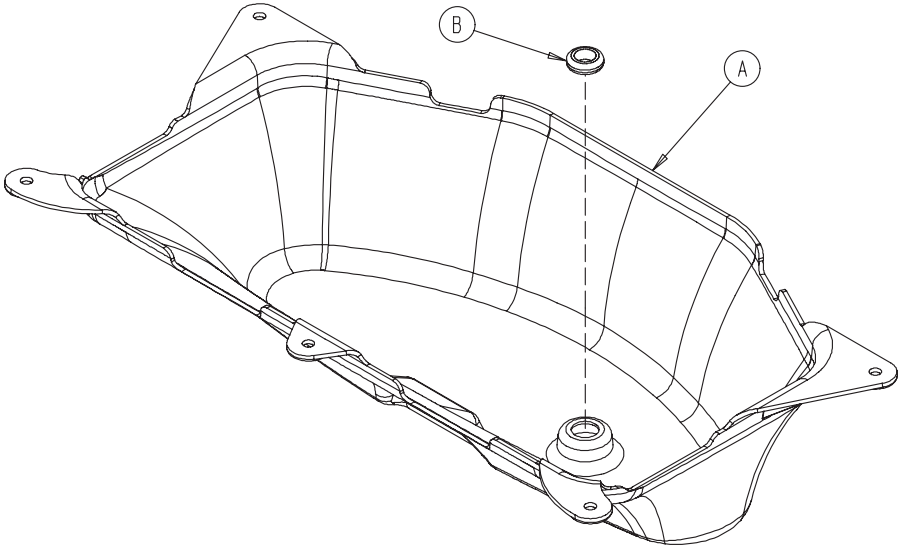


A	3002-001-100	1
B	3000-000-039	1

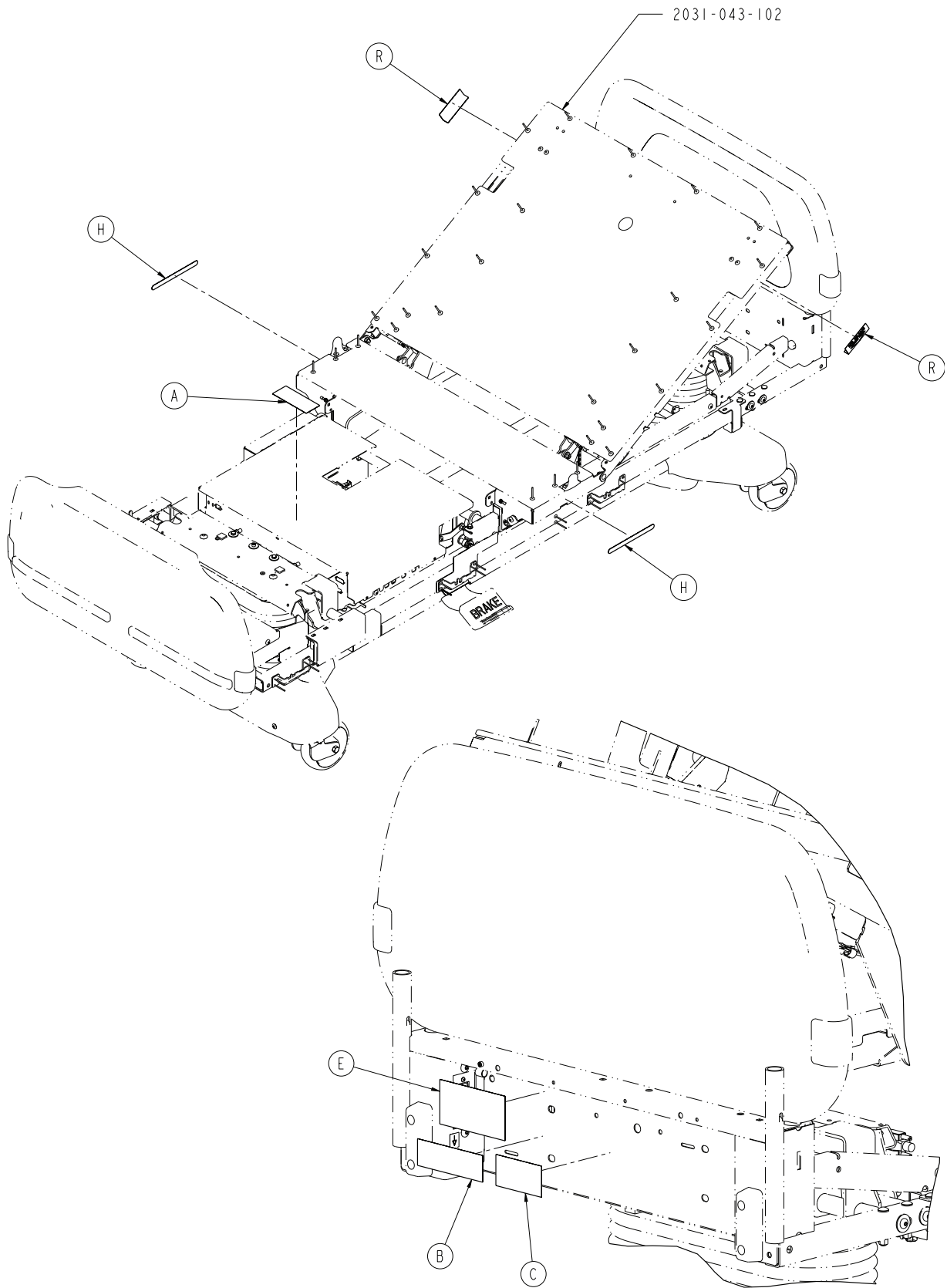


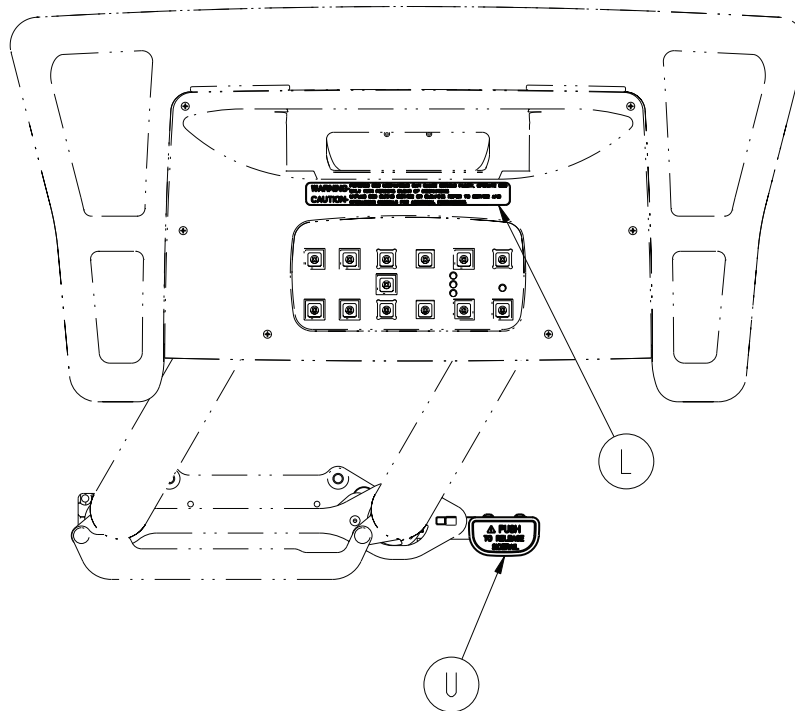
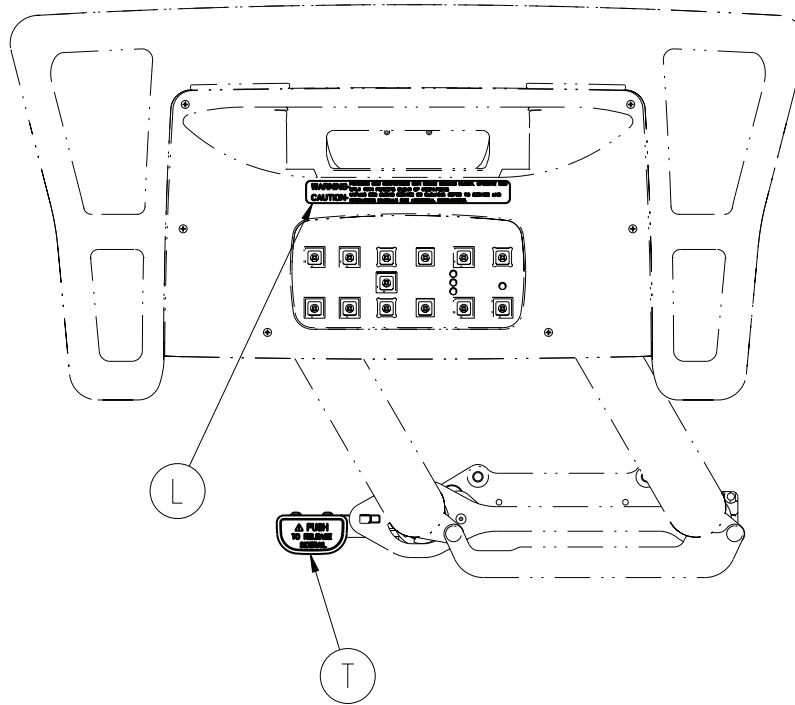



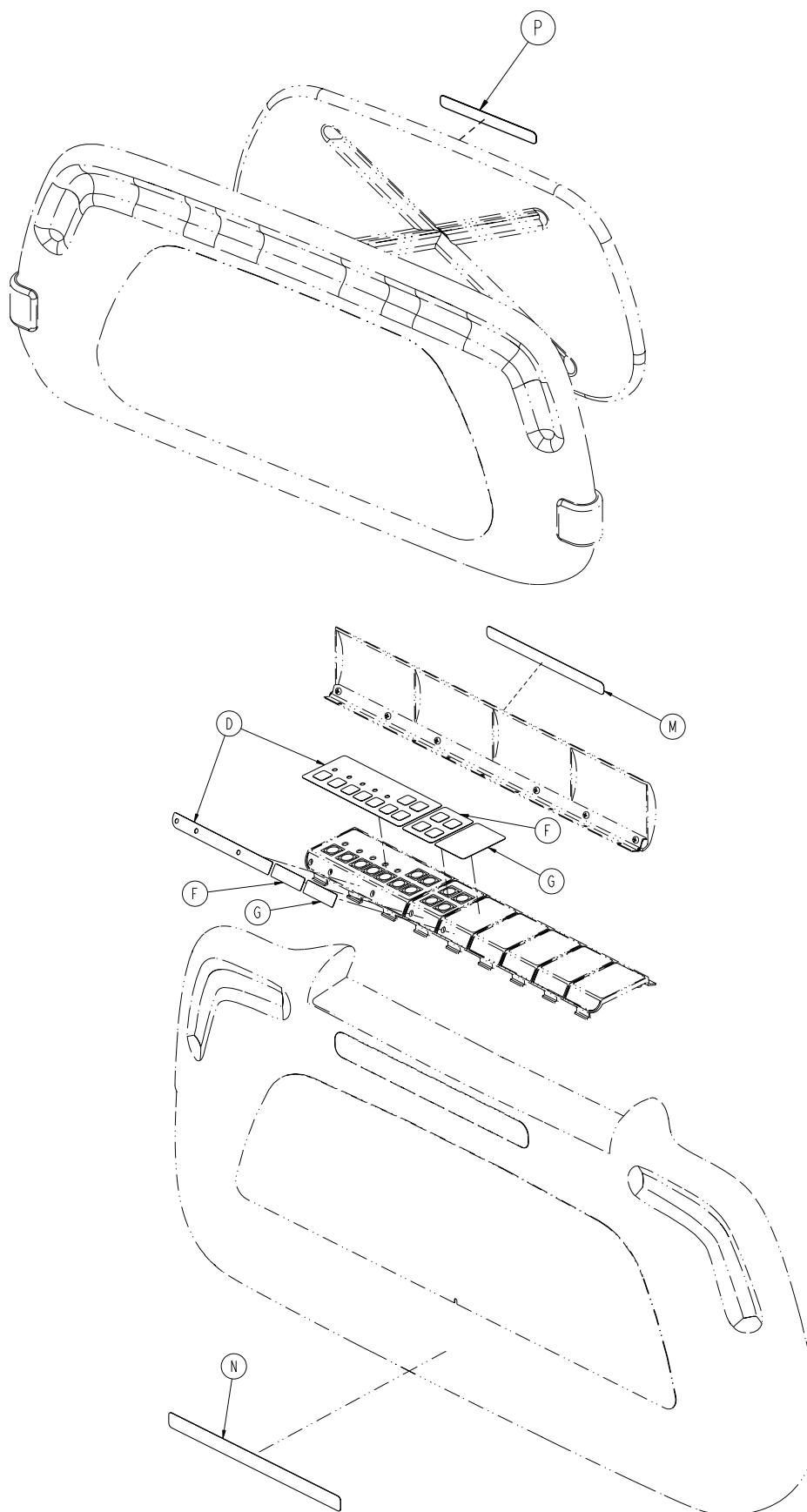



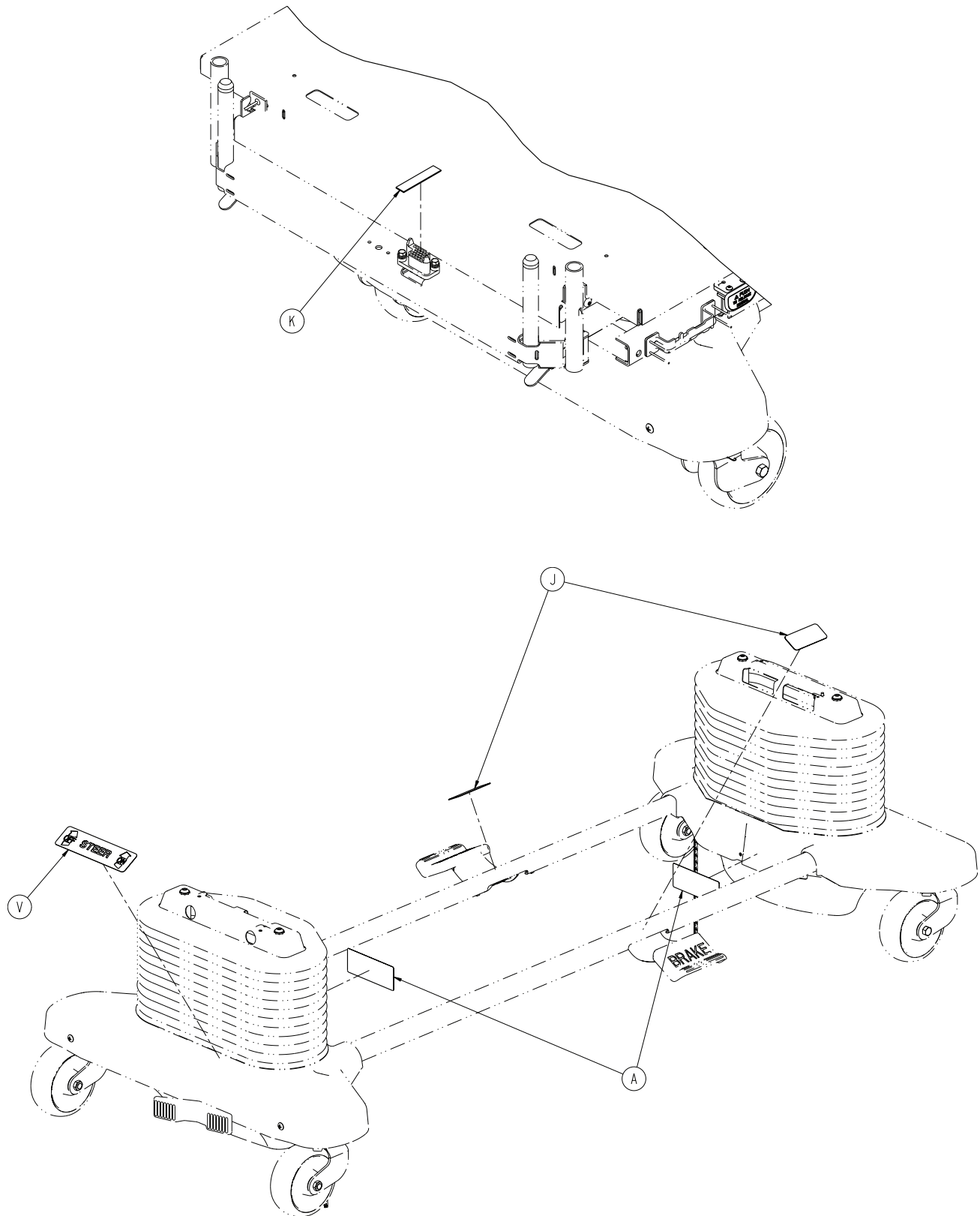


A	0988-002-708 (English)	
	2031-006-002 (Polish)	
	2031-008-002 (Portuguese)	3
B	1550-090-001	
	2031-006-003 (Polish)	
	2031-008-003 (Portuguese)	1
C	2011-001-104	
	2031-006-004 (Polish)	
	2031-008-004 (Portuguese)	1
D	2030-000-151	
	2031-006-005 (Polish)	
	2031-008-005 (Portuguese)	1
E	2031-231-128	
	2031-006-001 (Polish)	
	2031-008-001 (Portuguese)	1
F	2035-000-153	
	2031-006-006 (Polish)	
	2031-008-006 (Portuguese)	1
G	2035-000-155	
	2031-006-007 (Polish)	
	2031-008-007 (Portuguese)	1
H	2040-090-100	
	2031-006-008 (Polish)	
	2031-008-008 (Portuguese)	2
J	3000-200-601	
	2031-006-009 (Polish)	
	2031-008-009 (Portuguese)	2
K	3000-300-641	
	2031-006-010 (Polish)	
	2031-008-010 (Portuguese)	1
L	3000-400-556	
	2031-006-011 (Polish)	
	2031-008-011 (Portuguese)	2
M	3000-500-025	
	2031-006-012 (Polish)	
	2031-008-012 (Portuguese)	1
N	3000-500-029	
	2031-006-013 (Polish)	
	2031-008-013 (Portuguese)	1
P	3000-526-003	
	2031-006-014 (Polish)	
	2031-008-014 (Portuguese)	1
R	3001-300-603	
	2031-006-015 (Polish)	
	2031-008-015 (Portuguese)	2
T	3002-400-055	
	2031-006-216 (Polish)	
	2031-008-216 (Portuguese)	2
U	3002-400-065	
	2031-006-217 (Polish)	
	2031-008-217 (Portuguese)	2
V	5000-090-013	
	2031-006-017 (Polish)	
	2031-008-017 (Portuguese)	1

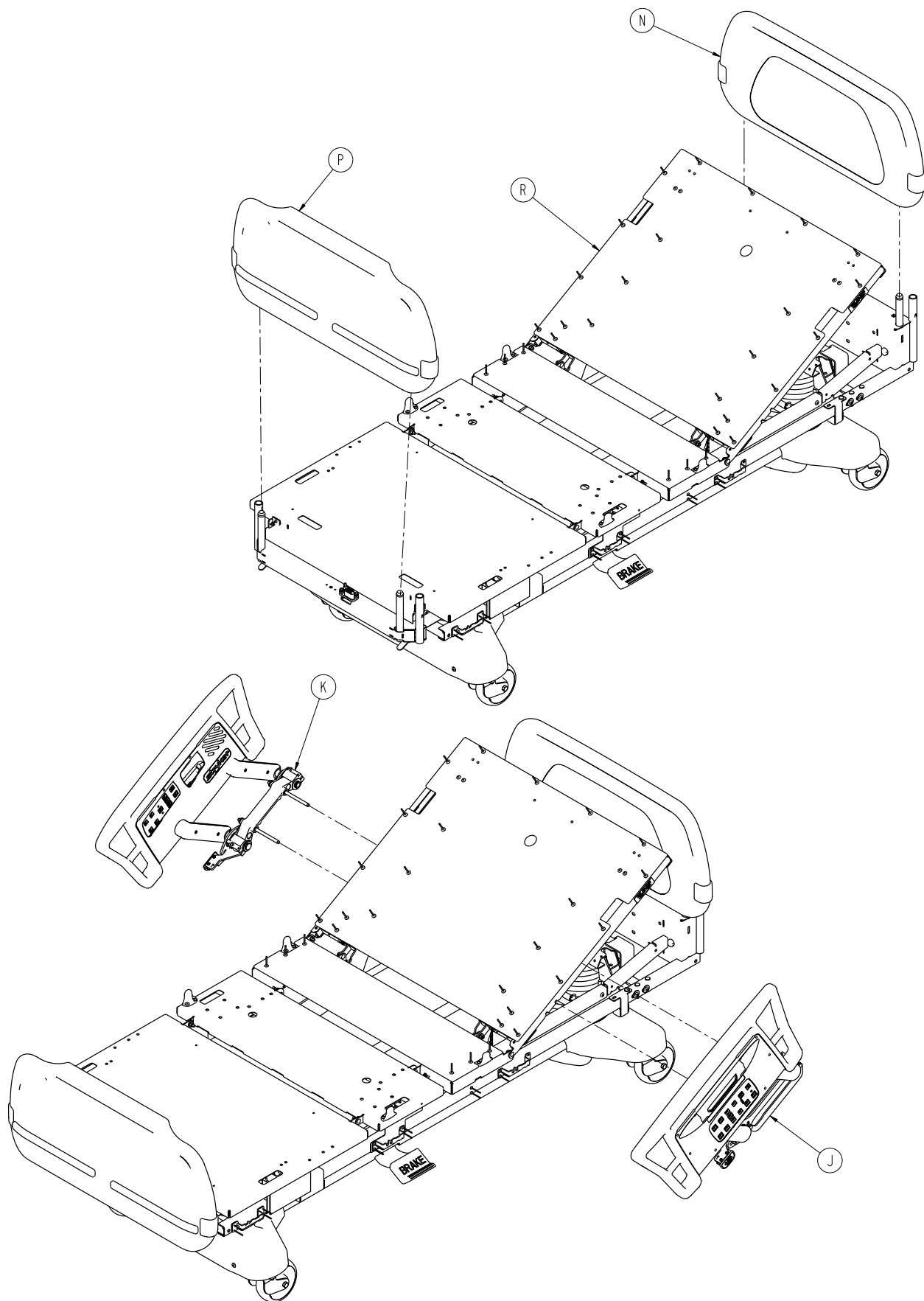


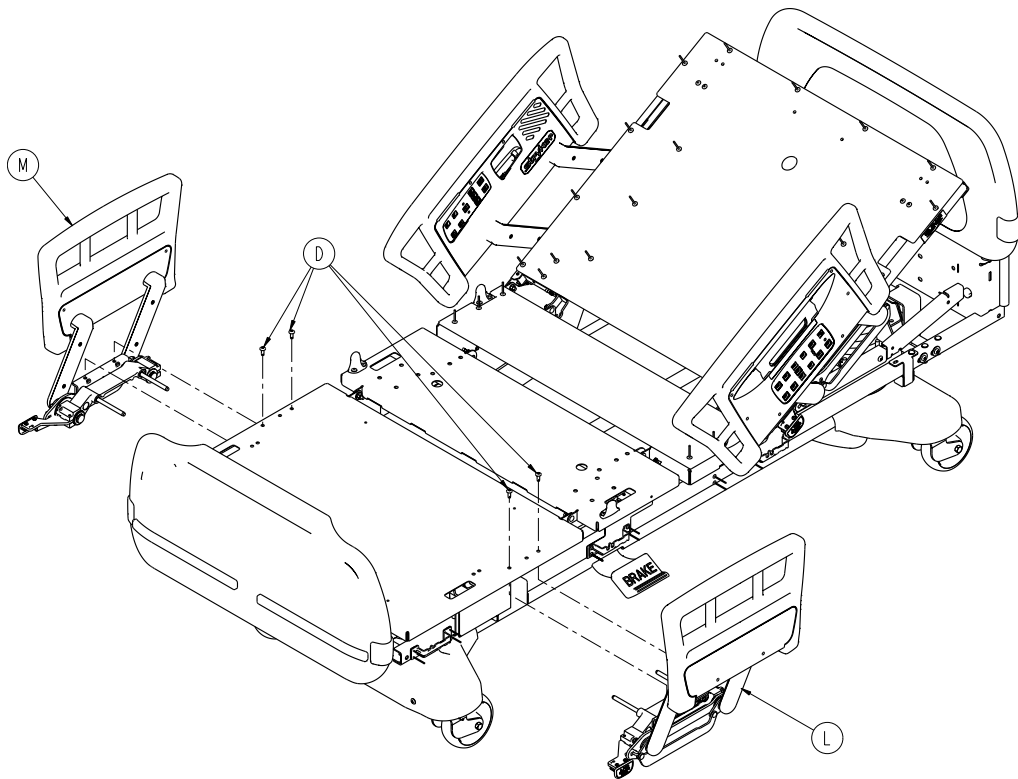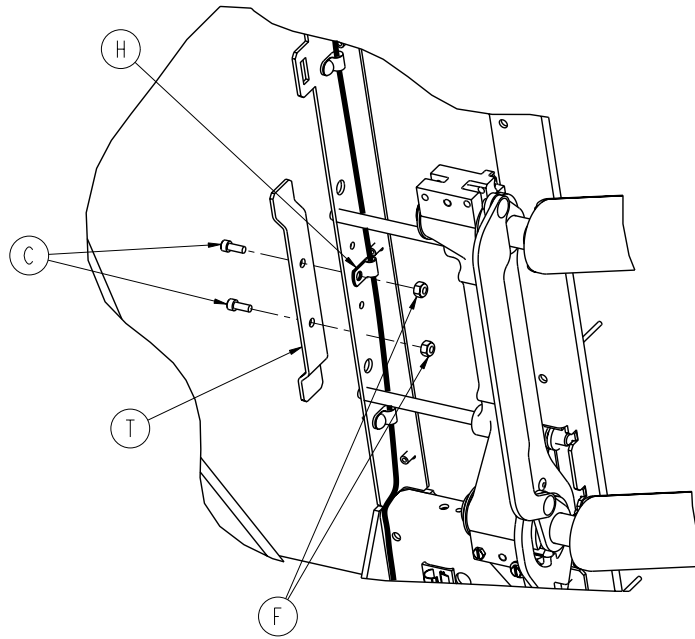


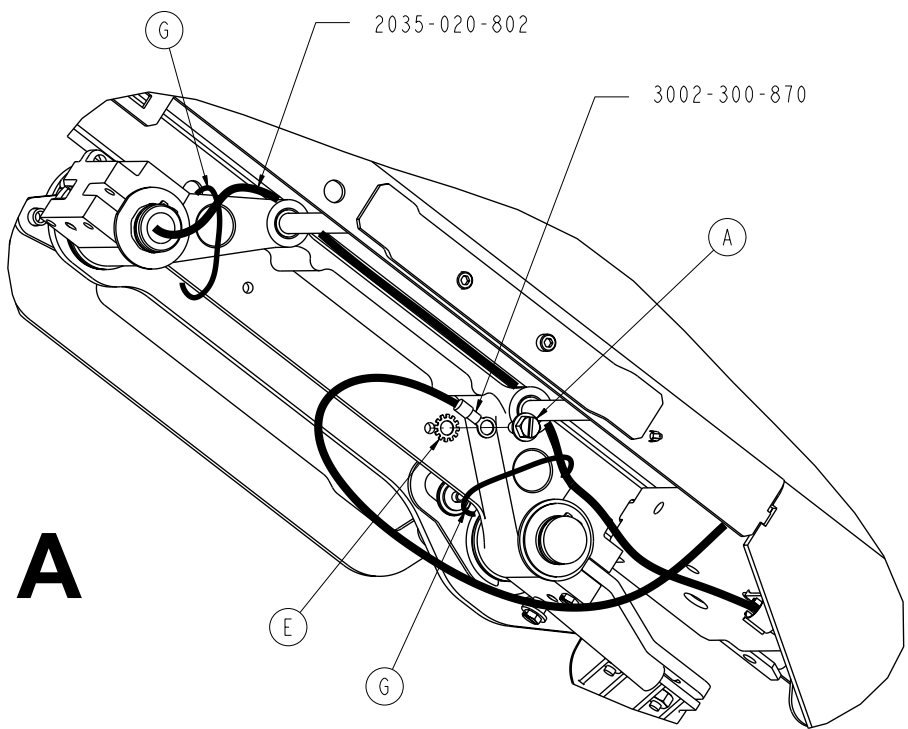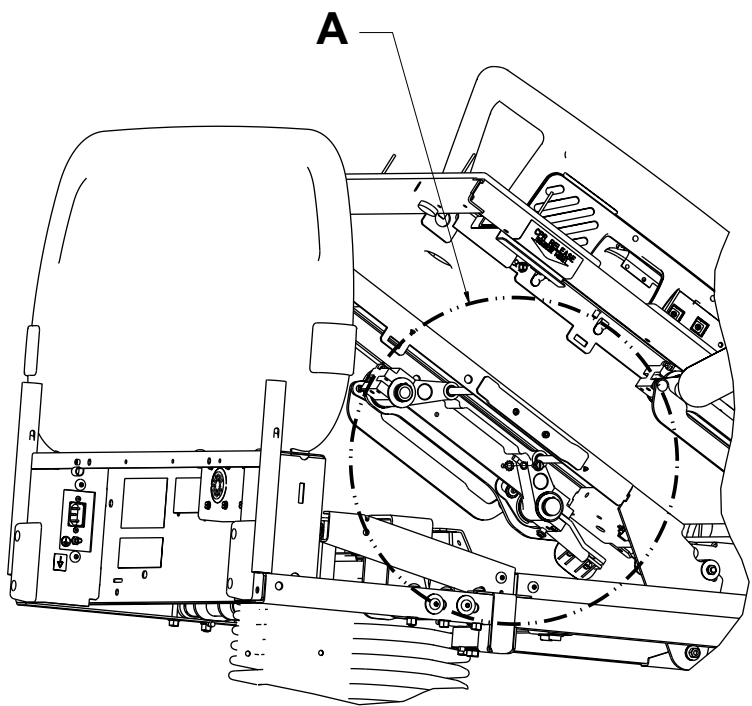

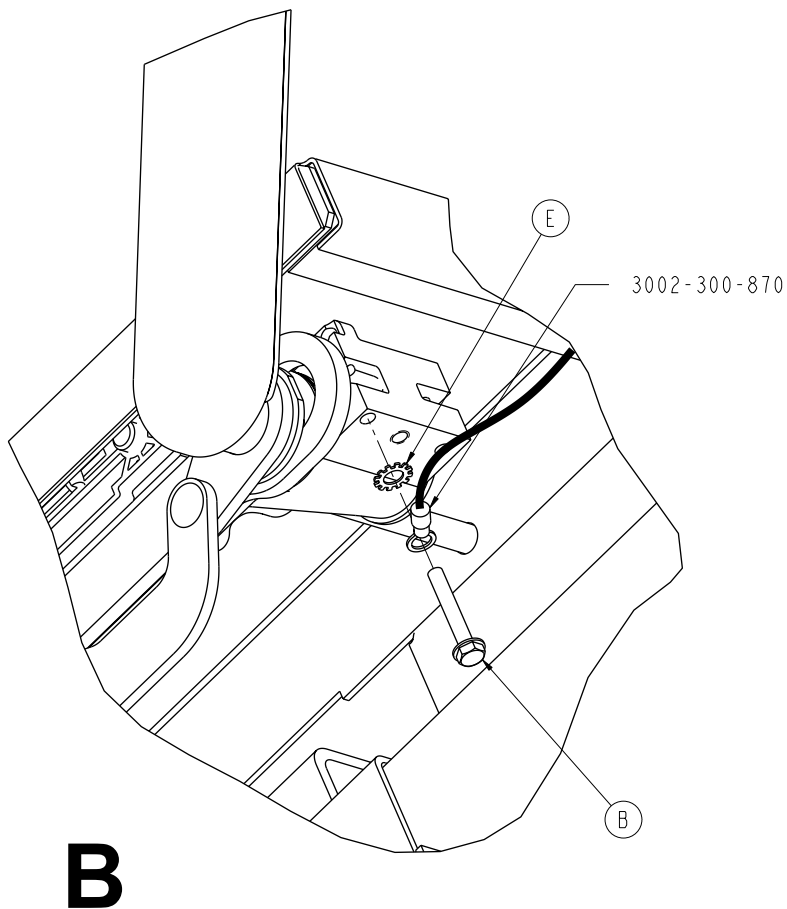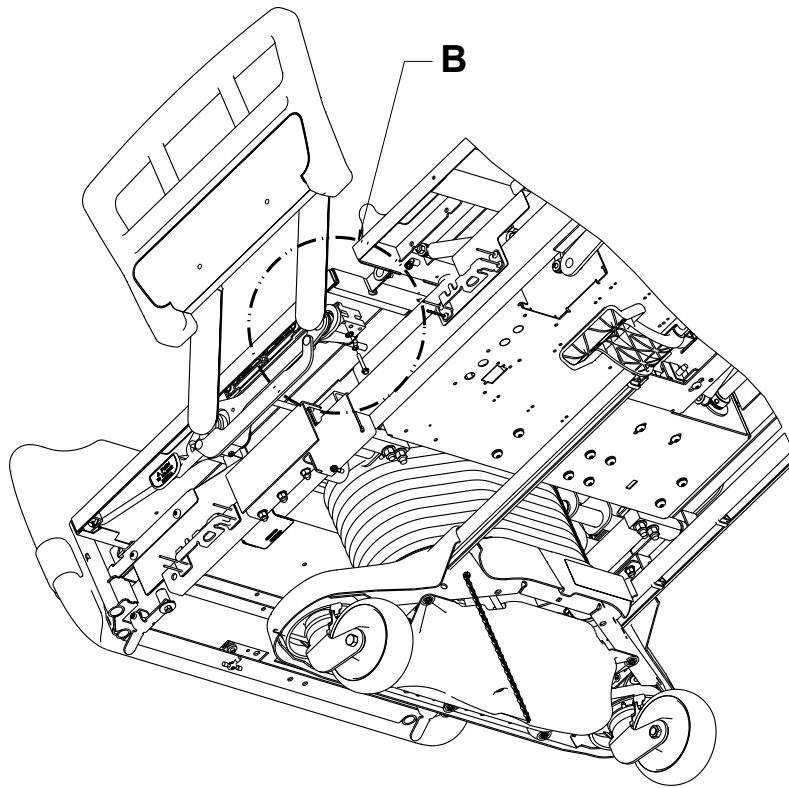



A	0003-221-000	2
B	0003-369-000	2
C	0004-032-000	4
D	0007-065-000	4
E	0013-010-000	4
F	0016-003-000	4
G	0038-151-000	4
H	0059-767-000	2
J	2031-400-105	1
K	2031-400-205	1
L	2031-400-305	1
M	2031-400-405	1
N	2032-130-010	1
P	2032-135-010	1
R	2032-300-010	1
T	2035-400-565	2

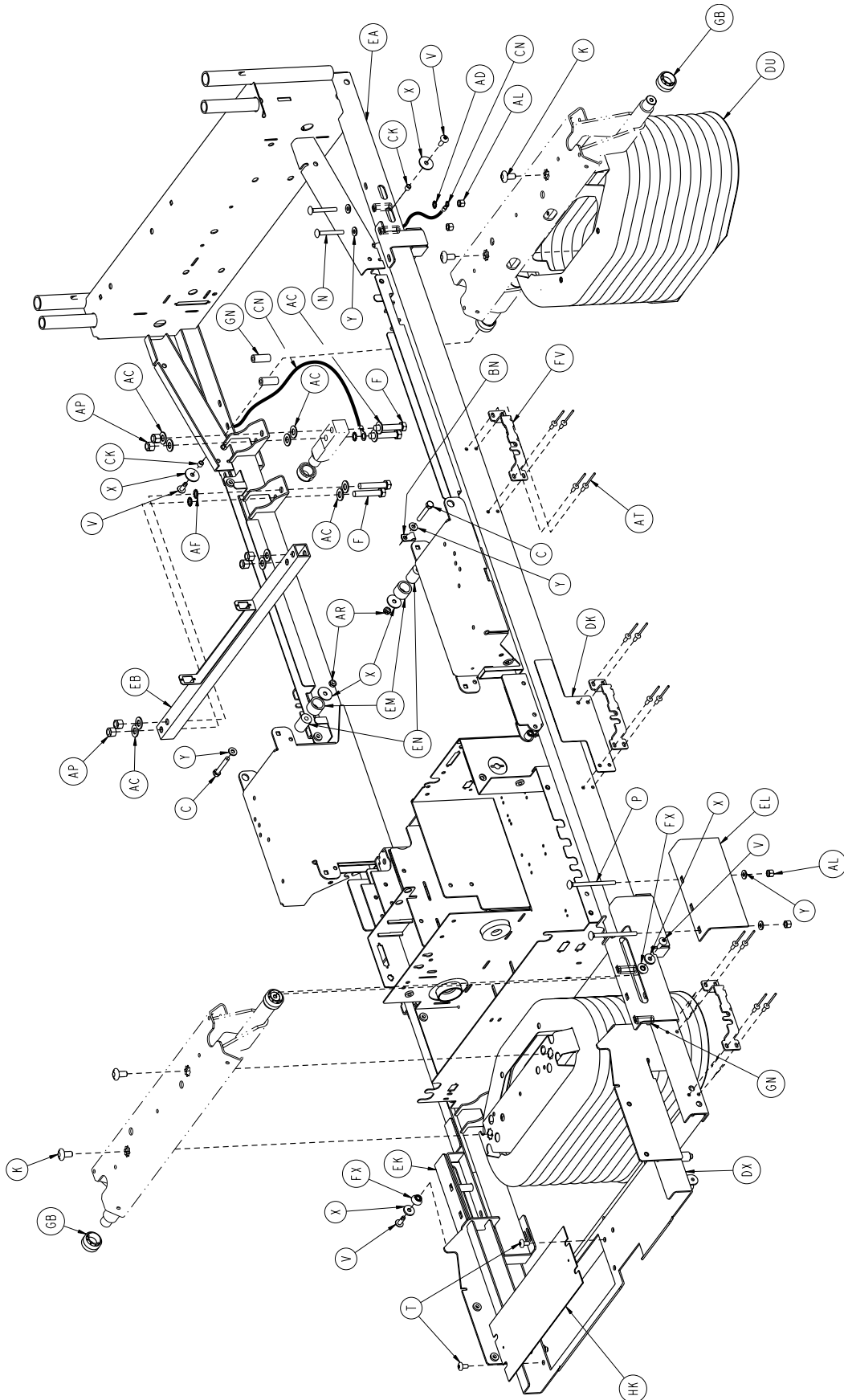


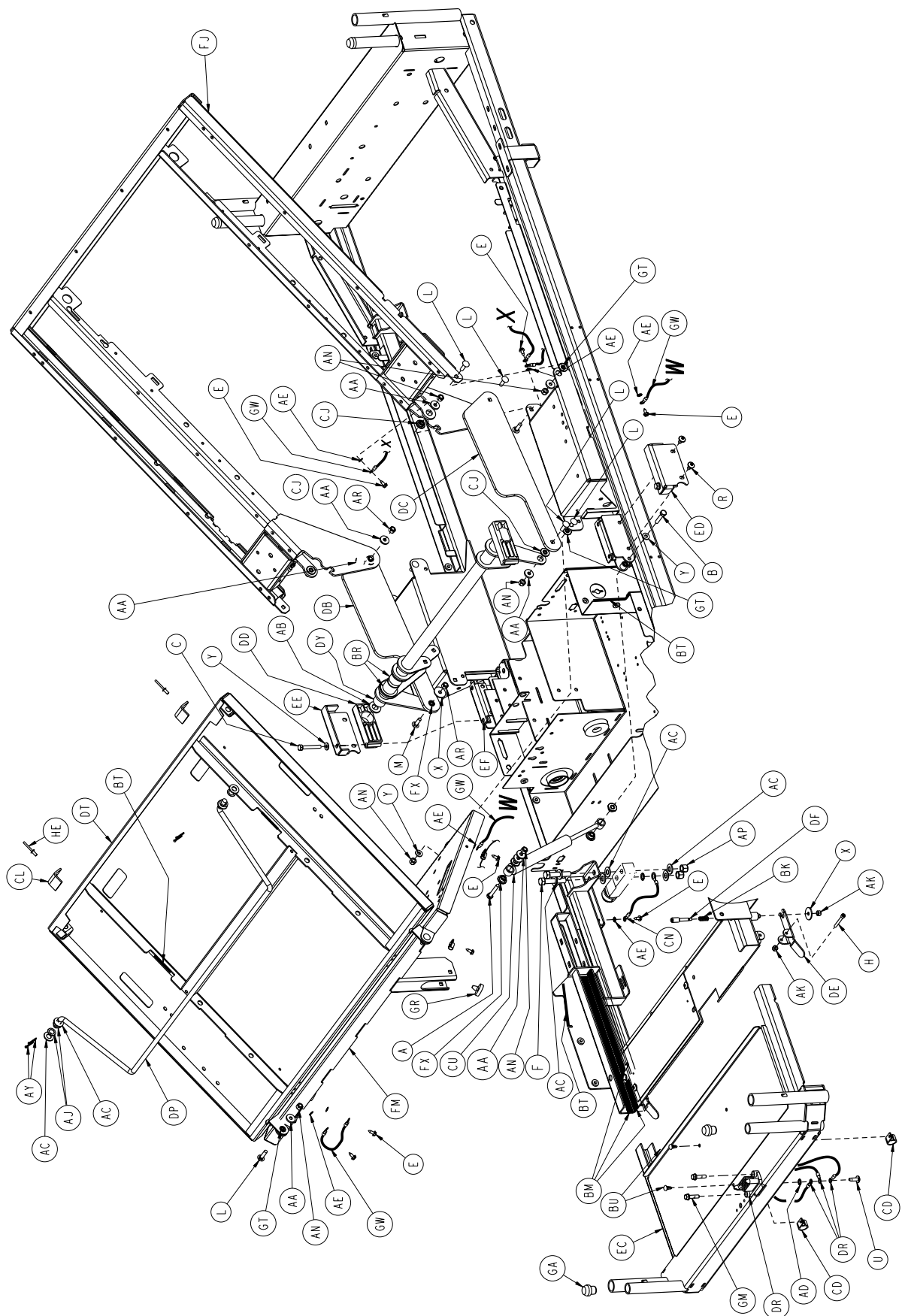



[Return To Table of Contents](#)

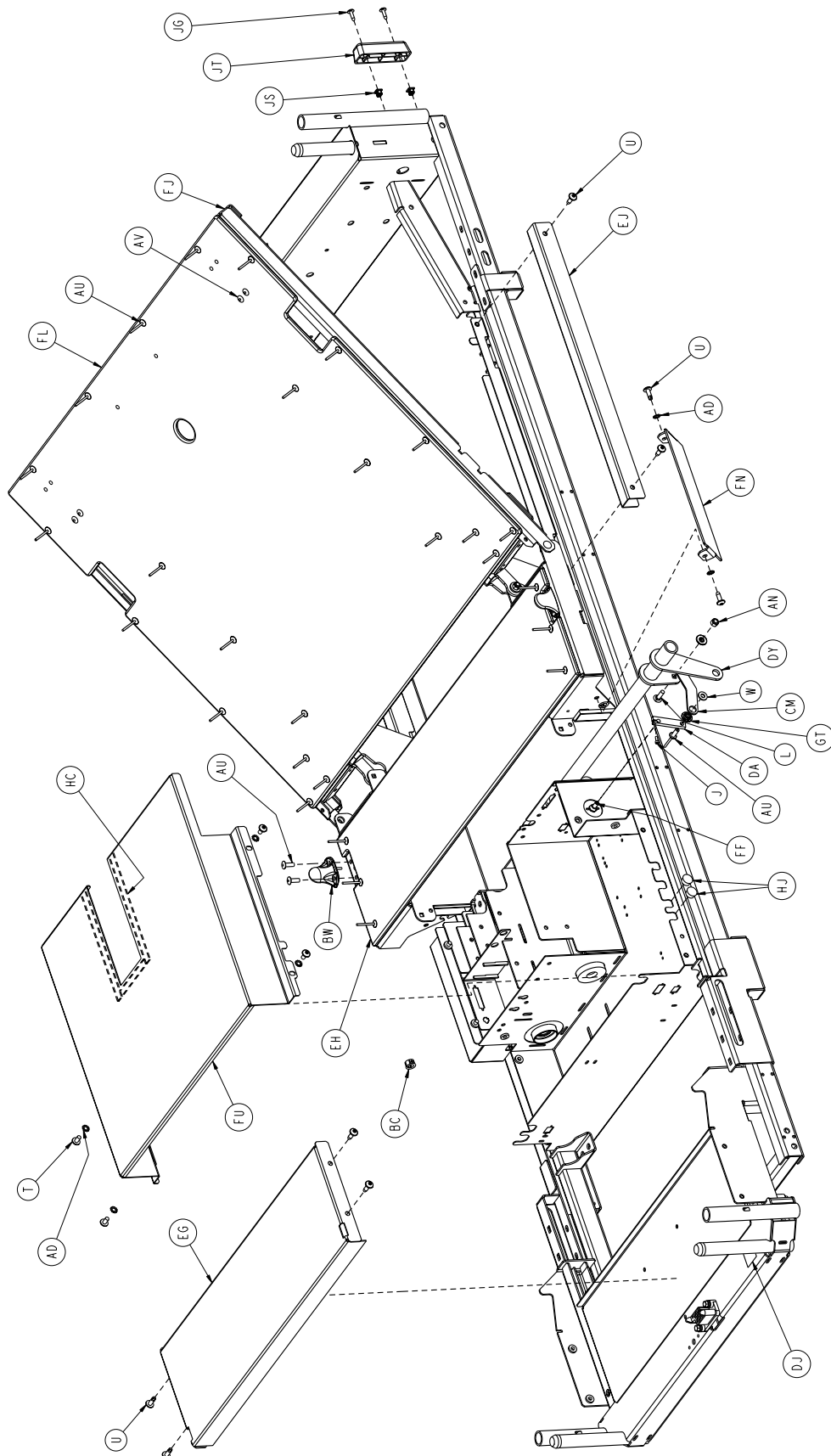


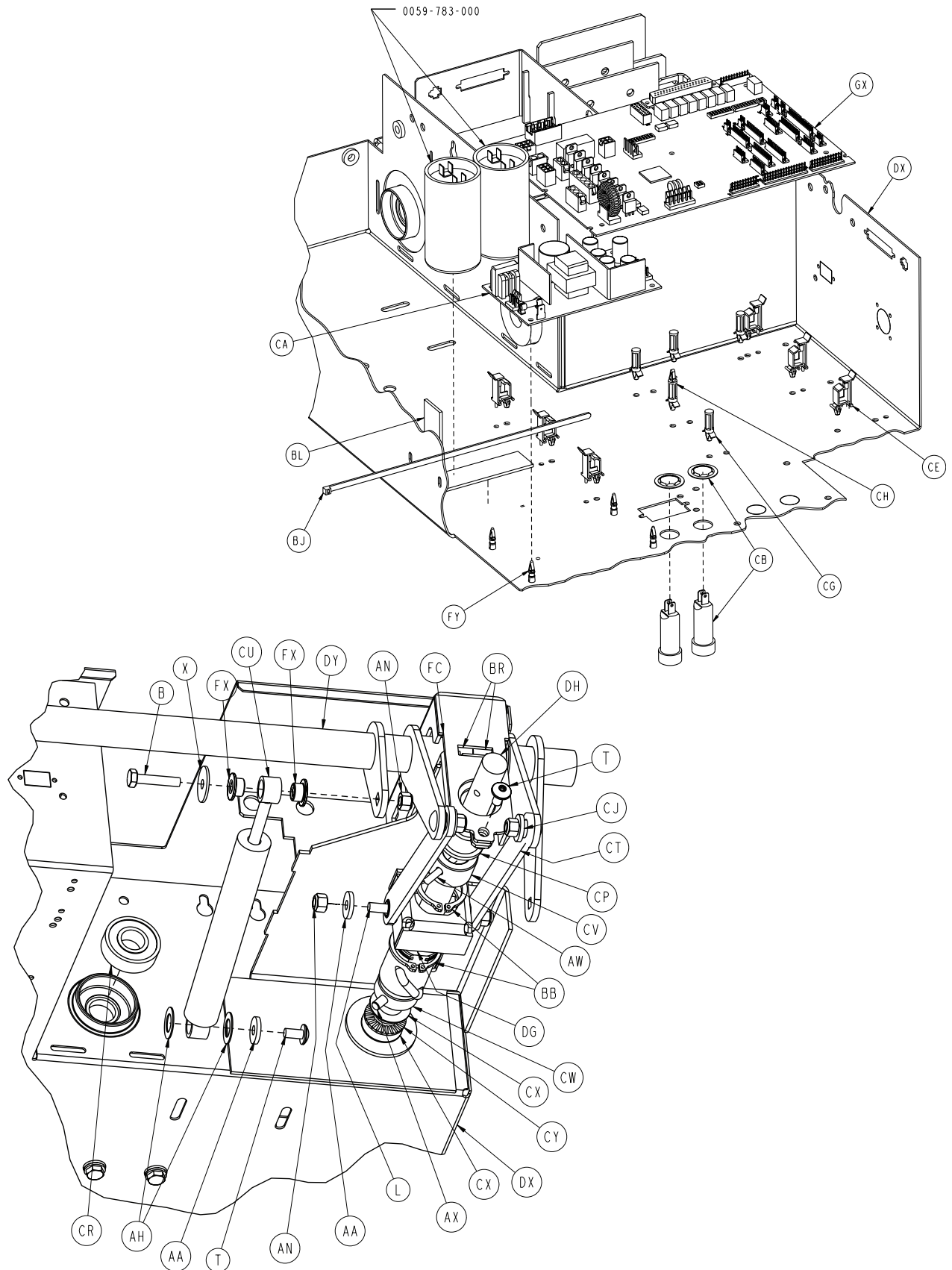


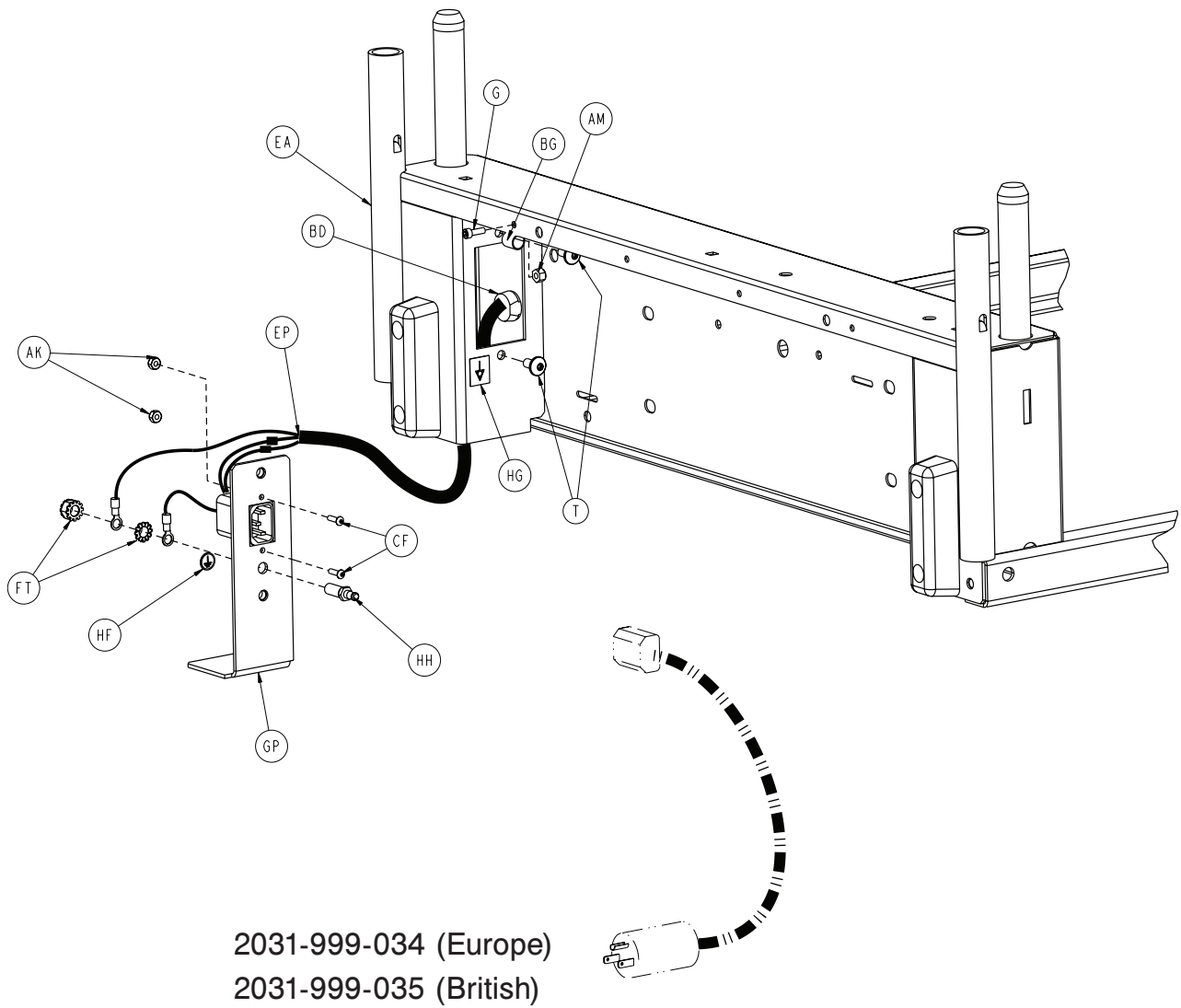

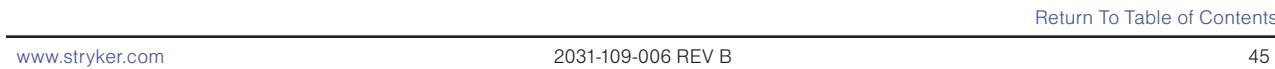

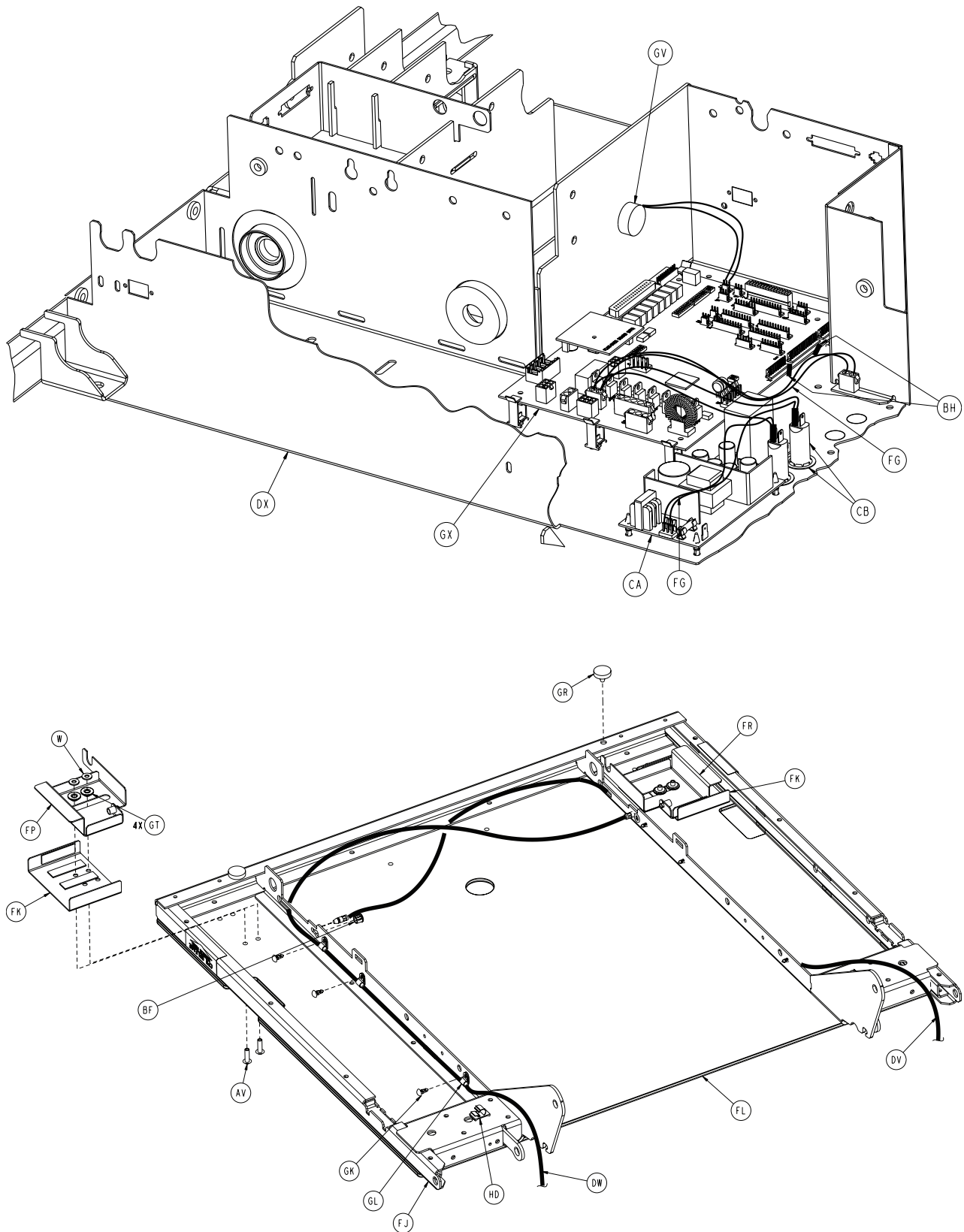

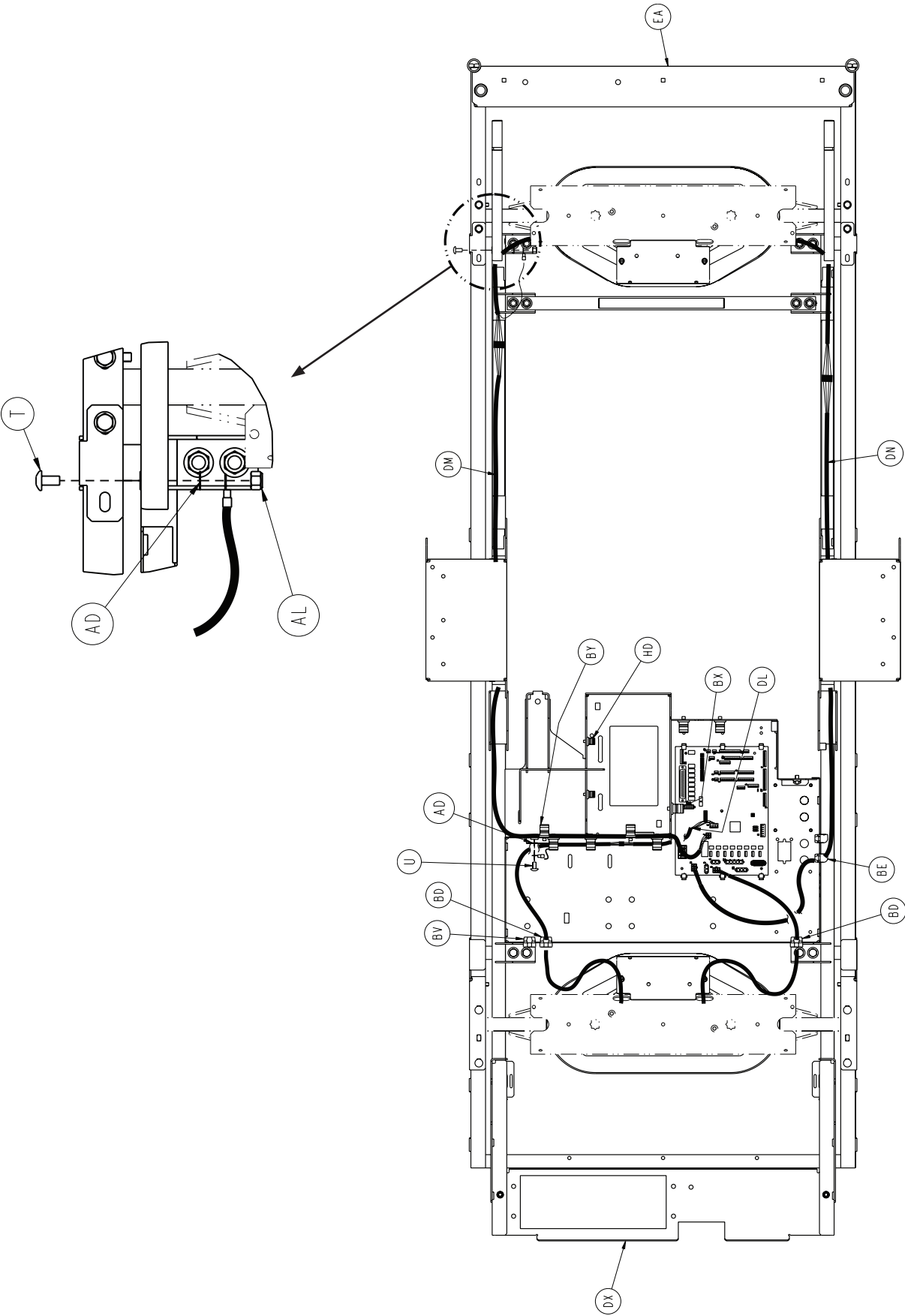

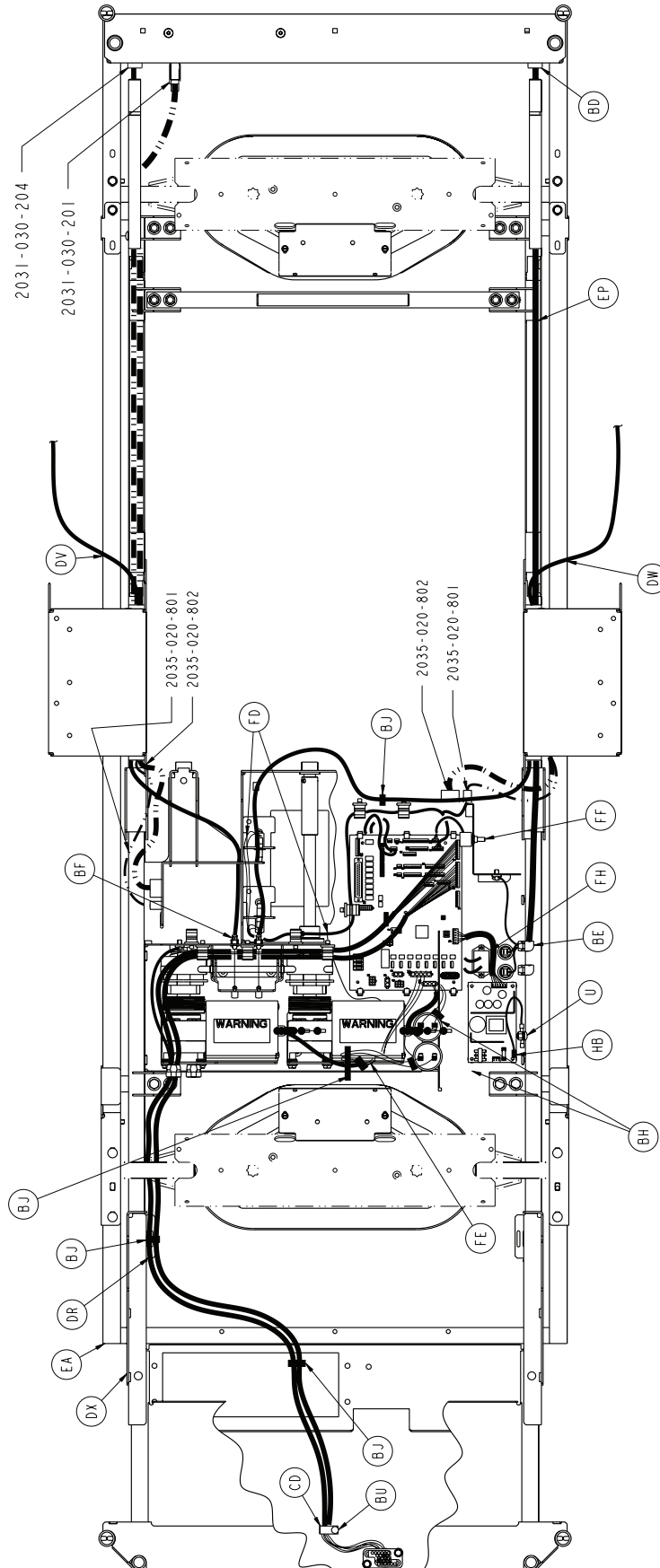

2032-300-010


A	0003-005-000	1	BM	0044-032-000	50"
B	0003-074-000	2	BN	0052-104-000	1
C	0003-078-000	4	BP	0052-759-000	2
D	0003-214-000	8	BR	0052-762-000	2
E	0003-224-000	16	BT	0058-056-000	2'
F	0003-347-000	12	BU	0058-076-000	2
G	0004-005-000	1	BV	0059-106-000	1
H	0004-085-000	2	BW	3003-300-033	4
J	0004-101-000	1	BX	0059-133-000	1
K	0004-338-000	4	BY	0059-135-000	8
L	0005-019-000	18	CA	0059-157-000	1
M	0005-023-000	3	CB	0059-179-000	2
N	0005-024-000	4	CC	0059-194-000	1
P	0005-029-000	4	CD	0059-743-000	2
R	0007-052-000	4	CE	0059-751-000	6
T	0007-058-000	10	CF	0004-126-000	2
U	0007-063-000	19	CG	0059-773-000	4
V	0007-065-000	12	CH	0059-774-000	1
W	0011-002-000	5	CJ	0081-268-000	13
X	0011-053-000	10	CK	0715-001-333	18
Y	0011-063-000	39	CL	2020-034-758	2
AA	0011-158-000	22	CM	2025-031-062	1
AB	0011-310-000	1	CN	2025-031-088	5
AC	0011-539-000	36	CP	2025-032-068	2
AD	0013-010-000	15	CR	2025-032-076	2
AE	0013-018-000	14	CT	2025-032-077	2
AF	0013-032-000	4	CU	2025-032-082	2
AG	0013-038-000	2	CV	2025-032-084	1
AH	0014-007-000	2	CW	2025-032-085	1
AJ	0014-008-000	4	CX	2025-032-086	5
AK	0016-023-000	2	CY	2025-032-087	2
AL	0016-006-000	9	DA	2025-231-061	1
AM	0016-014-000	1	DB	2025-231-088	1
AN	0016-028-000	16	DC	2025-231-089	1
AP	0016-035-000	12	DD	2025-231-090	2
AR	0016-102-000	7	DE	2025-231-099	2
AT	0025-050-000	24	DF	2025-231-112	2
AU	0025-142-000	37	DG	2025-232-089	1
AV	0025-147-000	4	DH	2025-232-090	1
AW	0026-012-000	2	DJ	2030-009-901	1
AX	0026-168-000	2	DK	2030-031-148	1
AY	0027-015-000	4	DL	2030-031-801	1
BA	0027-017-000	2	DM	2030-031-802	1
BB	0028-120-000	3	DN	2030-031-803	1
BC	0030-036-000	4	DP	2030-035-096	1
BD	0030-045-000	4	DR	2030-049-801	1
BE	0030-047-000	2	DT	2032-035-150	1
BF	0030-052-000	4	DU	2032-200-003	1
BG	0034-021-000	1	DV	2035-031-048	1
BH	0038-111-000	10	DW	2035-031-049	1
BJ	0038-151-000	18	DX	2032-031-050	1
BK	0038-382-000	2	DY	2035-031-051	1
BL	0044-029-000	1	EA	2035-031-054	1

[Return To Table of Contents](#)

2032-300-010


EB	2035-031-055	1	GN	3001-300-004	8
EC	2035-031-057	1	GP	2035-231-070	1
ED	2035-031-064	1	GR	3001-300-008	4
EE	2035-031-065	1	GT	3001-300-099	12
EF	2035-031-066	2	GU	3001-300-663	10
EG	2035-031-094	1	GV	3001-508-870	1
EH	2035-031-097	1	GW	3002-300-870	8
EJ	2035-031-100	2	GX	3002-407-950	1
EK	2035-031-115	1	GY	3221-300-705	2
EL	2035-031-116	1	HA	5000-030-366	1
EM	2035-031-126	2	HB	5010-080-007	1
EN	2035-031-127	2	HC	8800-380-000	1.5'
EP	2035-031-801	1	HD	8815-001-100	4
ER	2035-032-052	1	HE	0025-079-000	2
ET	2035-032-054	1	HF	0036-046-000	1
EU	2035-032-072	1	HG	0036-115-000	1
EV	2035-032-077	2	HH	2011-001-215	1
EW	2035-032-079	2	HJ	0037-085-000	2
EX	2035-032-084	1	HK	2035-031-203	1
EY	2035-032-085	1			
FA	2035-032-088	2			
FB	2035-032-090	1			
FC	2035-032-096	1			
FD	2035-032-801	1			
FE	2035-032-802	1			
FF	2035-032-803	1			
FG	2035-032-804	1			
FH	2035-032-805	1			
FJ	2032-033-050	1			
FK	2035-033-062	2			
FL	2035-033-063	1			
FM	2032-034-050	1			
FN	2035-231-085	2			
FP	2035-233-064	1			
FR	2035-233-065	1			
FT	0016-033-000	2			
FU	2035-432-075	1			
FV	2040-031-056	6			
FW	3000-300-058	2			
FX	3000-300-099	8			
FY	3000-300-115	4			
GA	3000-300-349	4			
GB	3000-300-353	4			
GC	3000-300-442	16			
GD	3000-300-455	4			
GE	3000-300-456	2			
GF	3000-300-461	2			
GG	3000-300-462	2			
GH	3000-300-464	4			
GJ	3000-300-473	2			
GK	3000-300-477	6			
GL	3000-300-478	6			
GM	3001-200-228	2			

2031-999-130

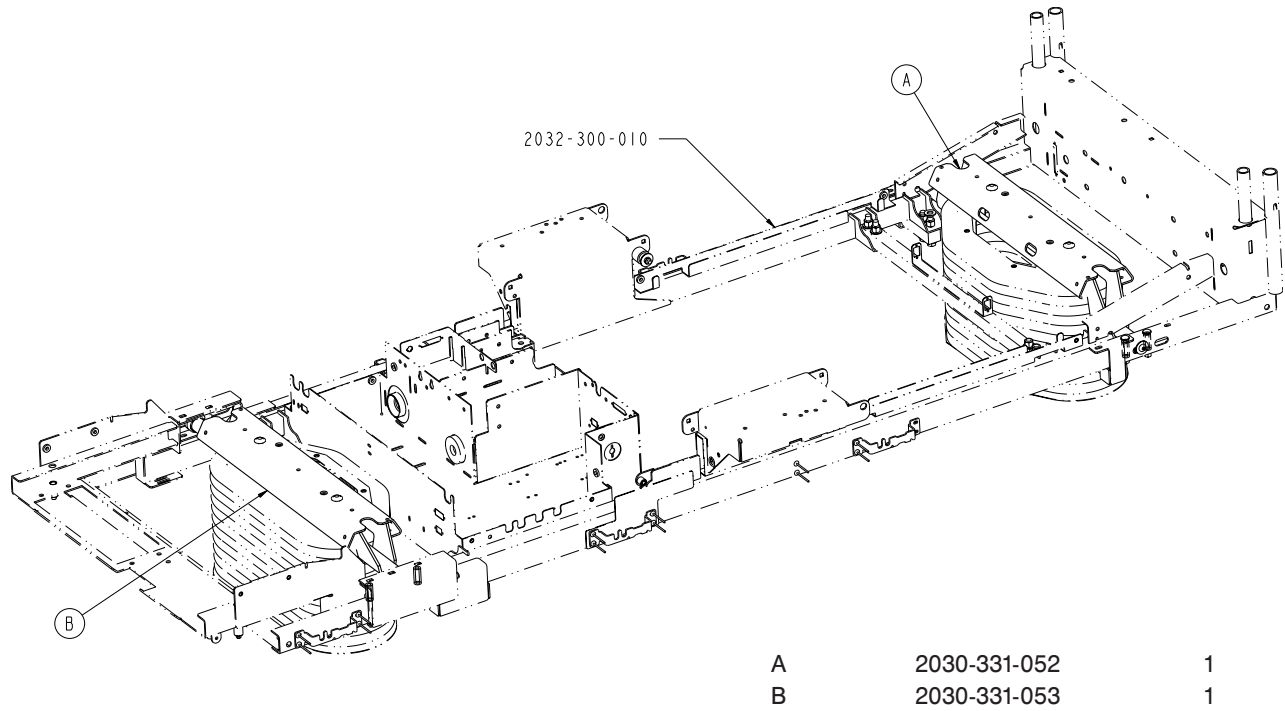


A	2030-331-052	1
B	2030-331-053	1

2031-999-131

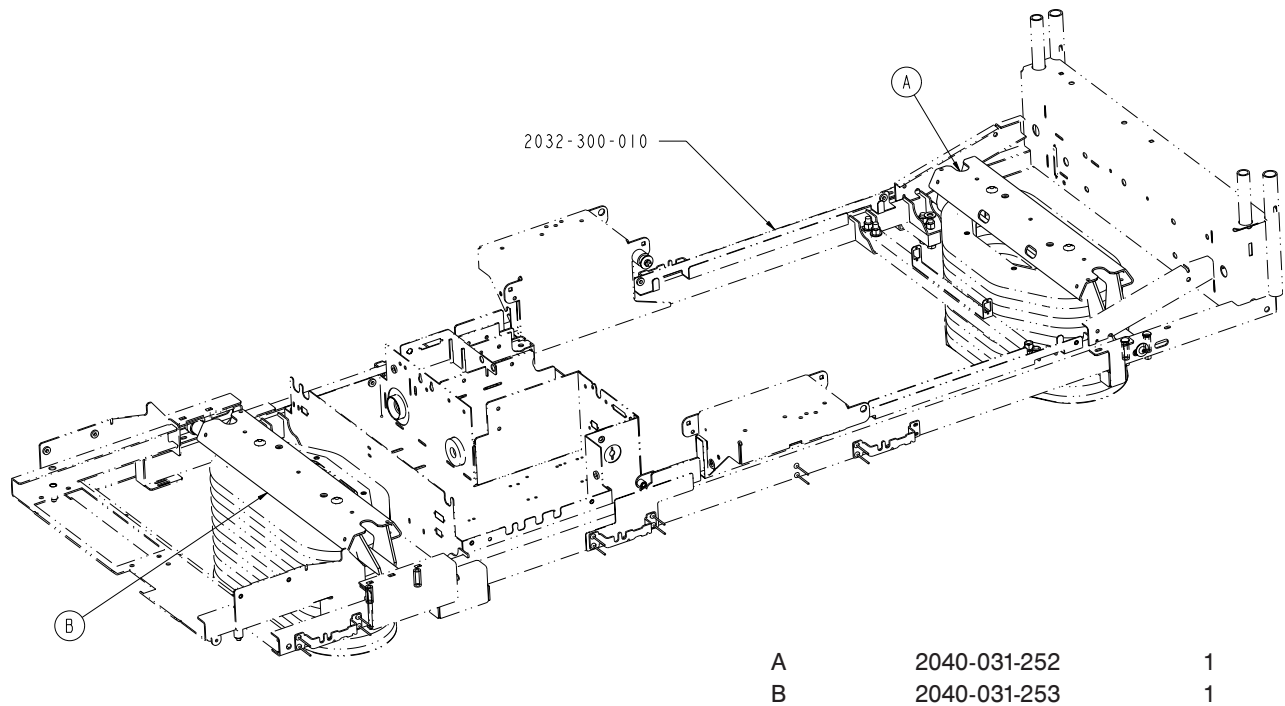



A	2040-031-252	1
B	2040-031-253	1

2031-999-034 (Europe)

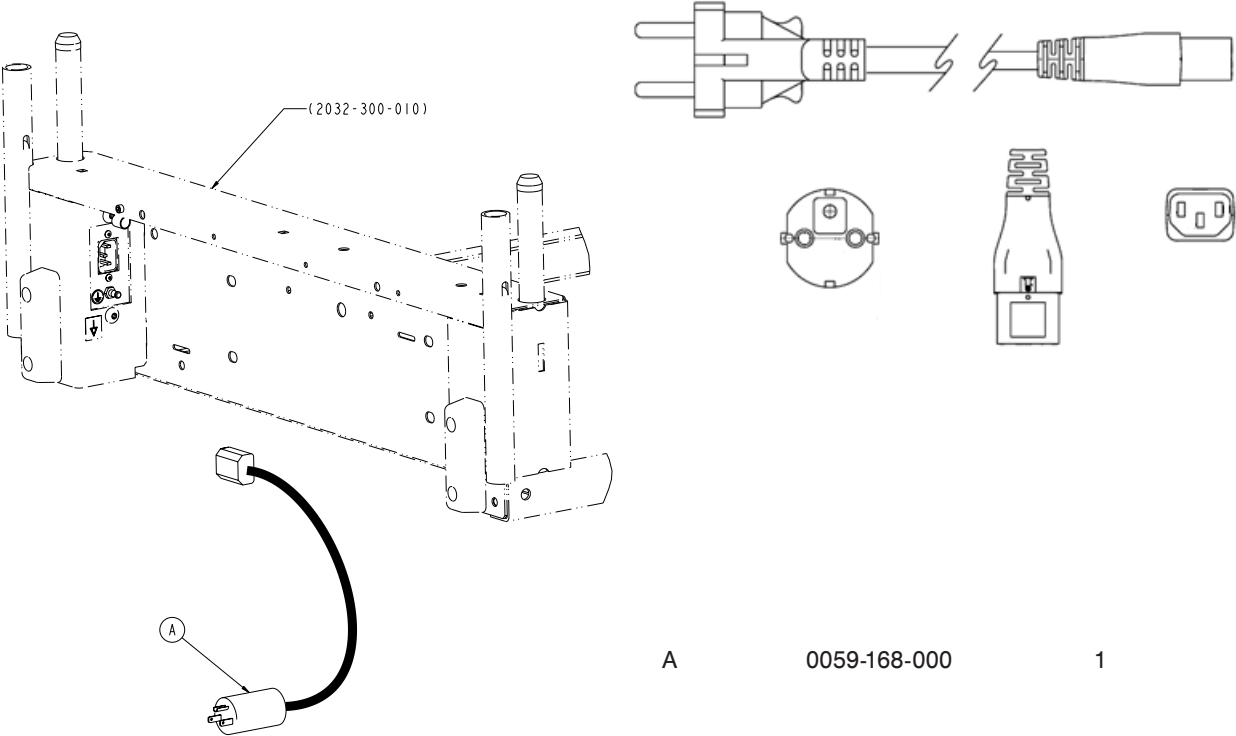


2031-999-035 (British)

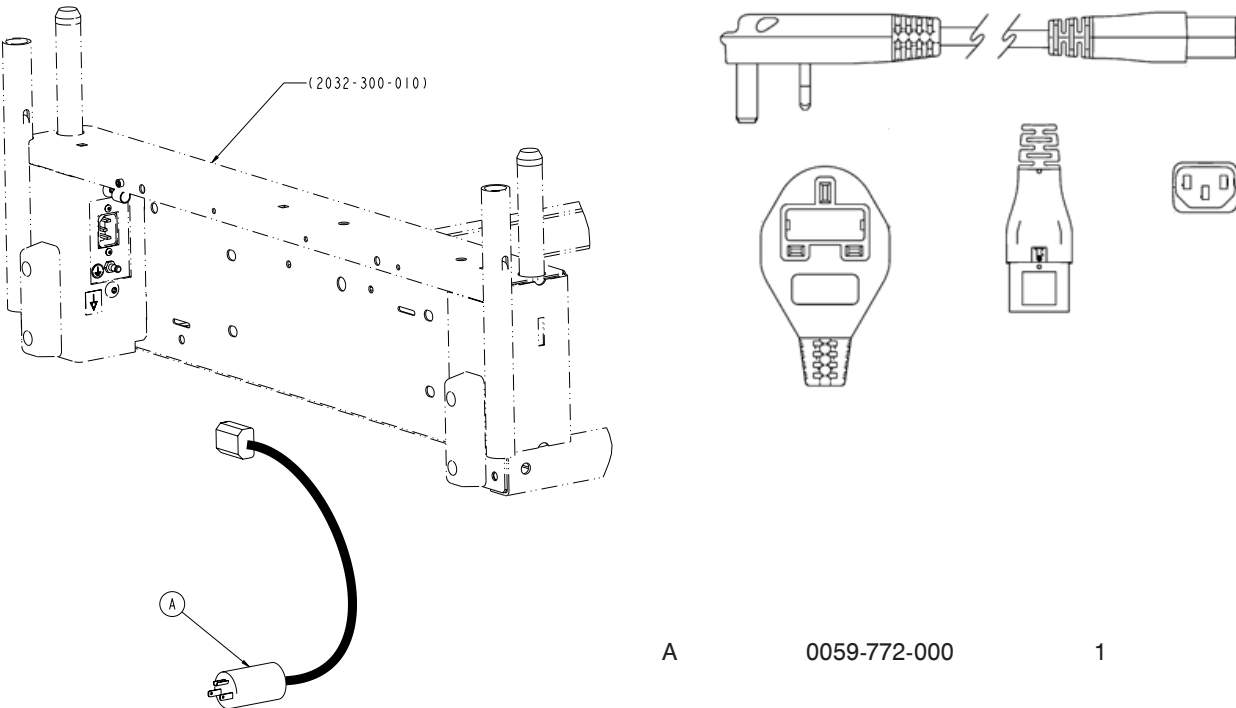


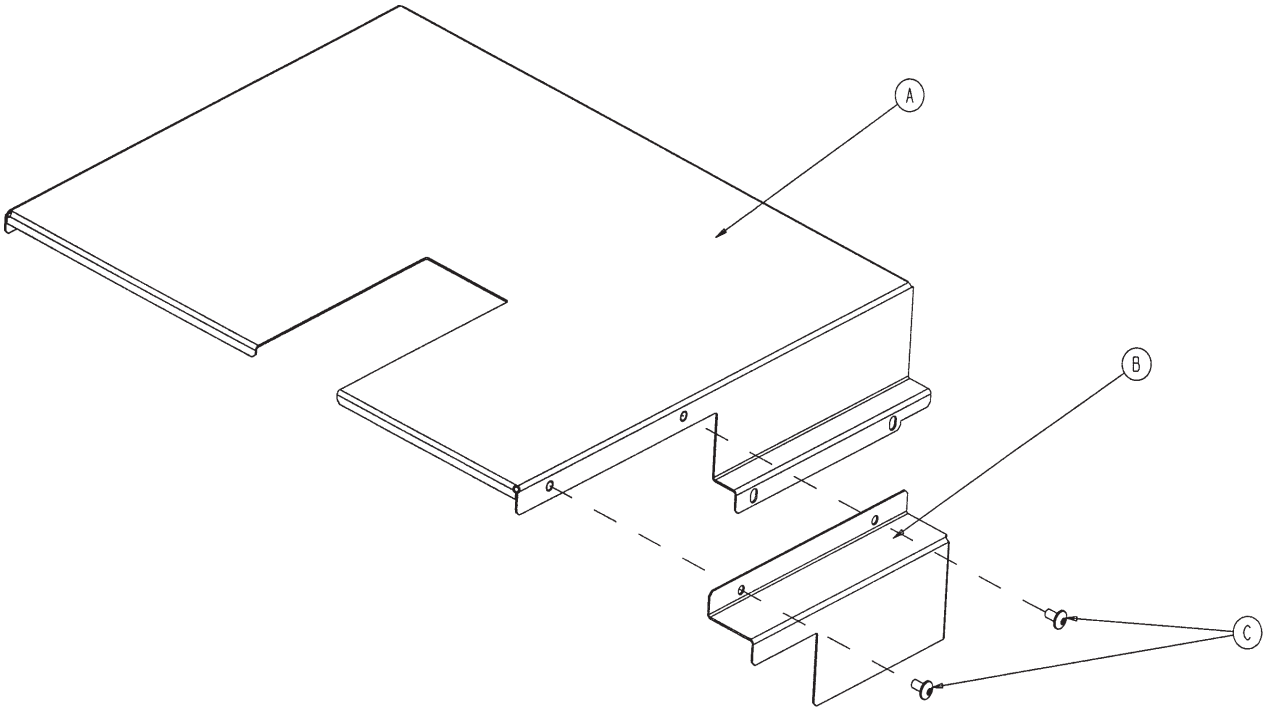


A	2035-332-075	1
B	2035-332-076	1
C	0007-058-000	2

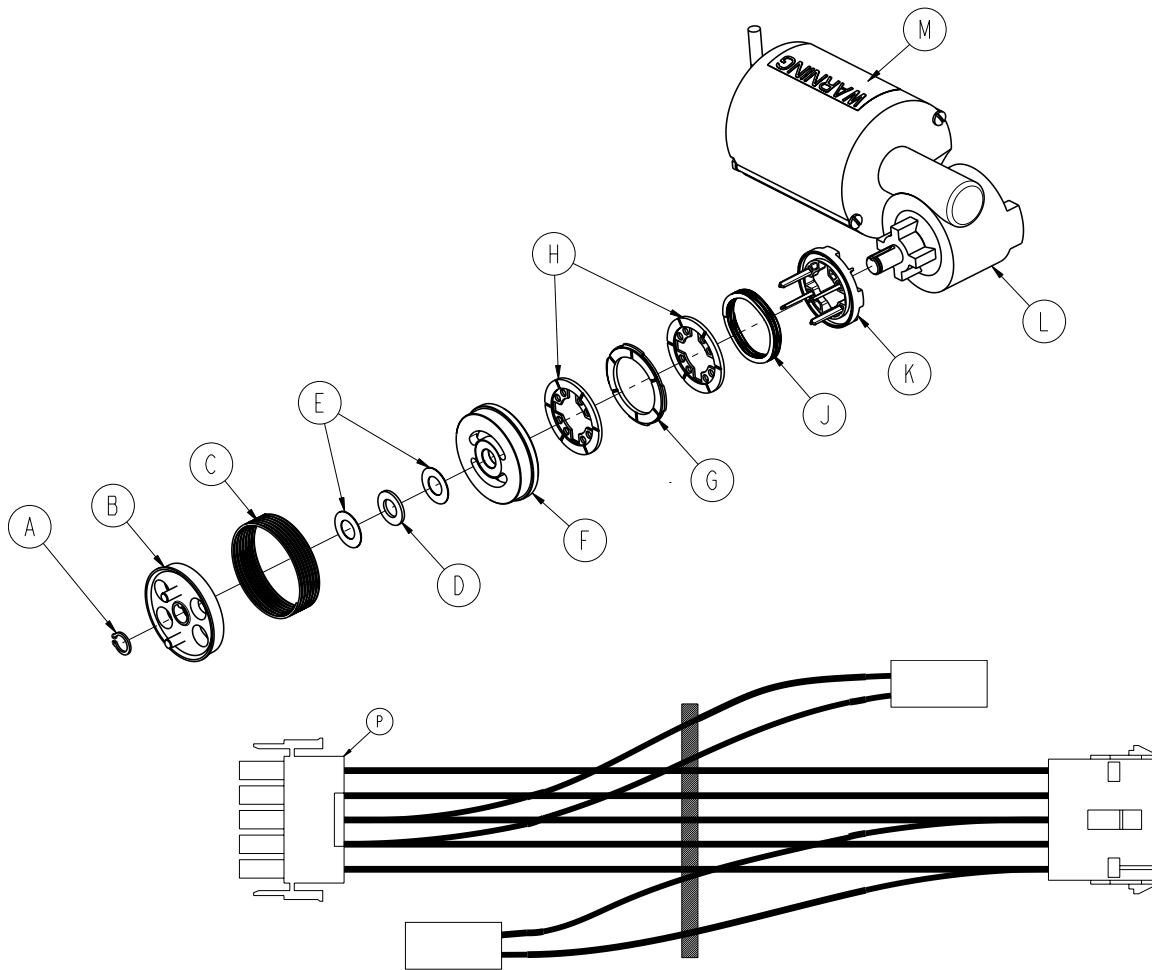


A	0028-131-000	1
B	3001-300-455	1
C	3000-300-465	1
D	0081-212-000	1
E	3000-200-224	2
F	3001-300-569	1
G	3001-300-552	1
H	3001-300-551	2
J	3001-300-563	1
K	3001-300-570	1
L	4712-116-004	1
M	3000-300-604	1
P	3221-300-271	1

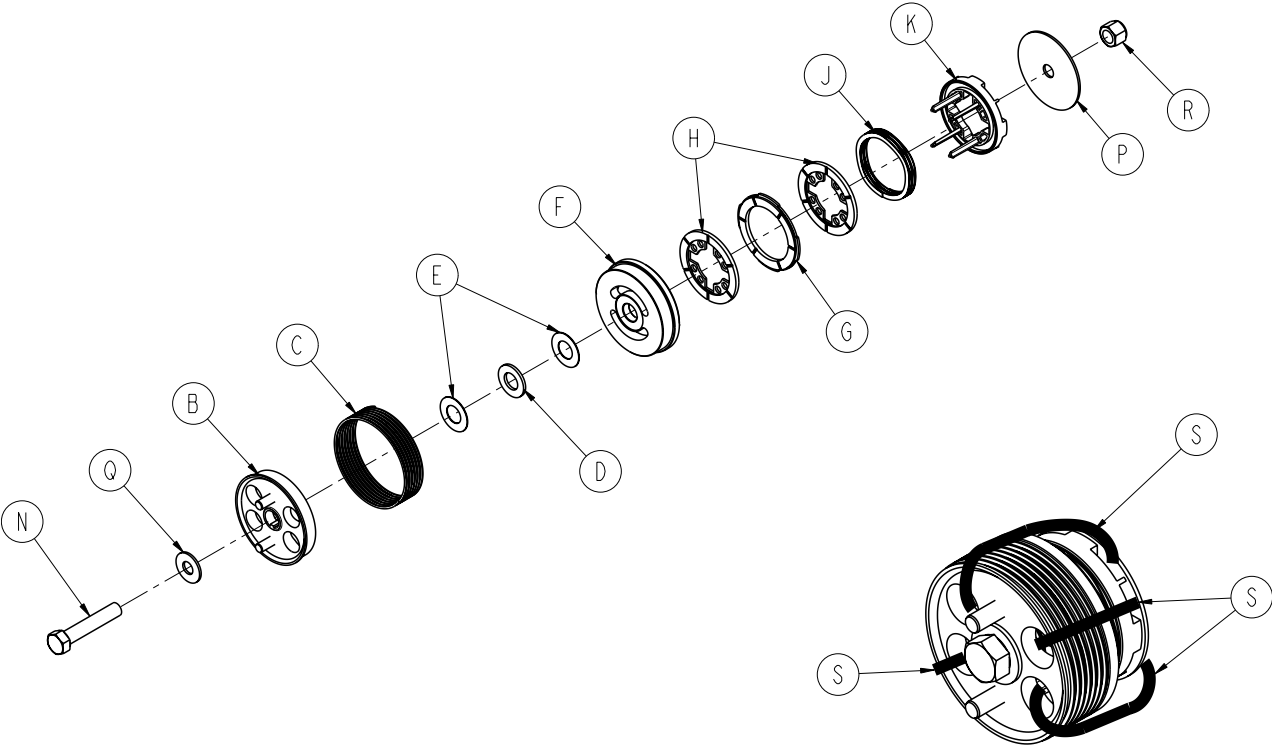


B	3001-300-455	1
C	3000-300-465	1
D	0081-212-000	1
E	3000-200-224	2
F	3001-300-569	1
G	3001-300-552	1
H	3001-300-551	2
J	3001-300-563	1
K	3001-300-570	1
N	0003-064-000	1
P	3001-300-776	1
Q	0011-193-000	1
R	0016-035-000	1
S	3000-300-113	4

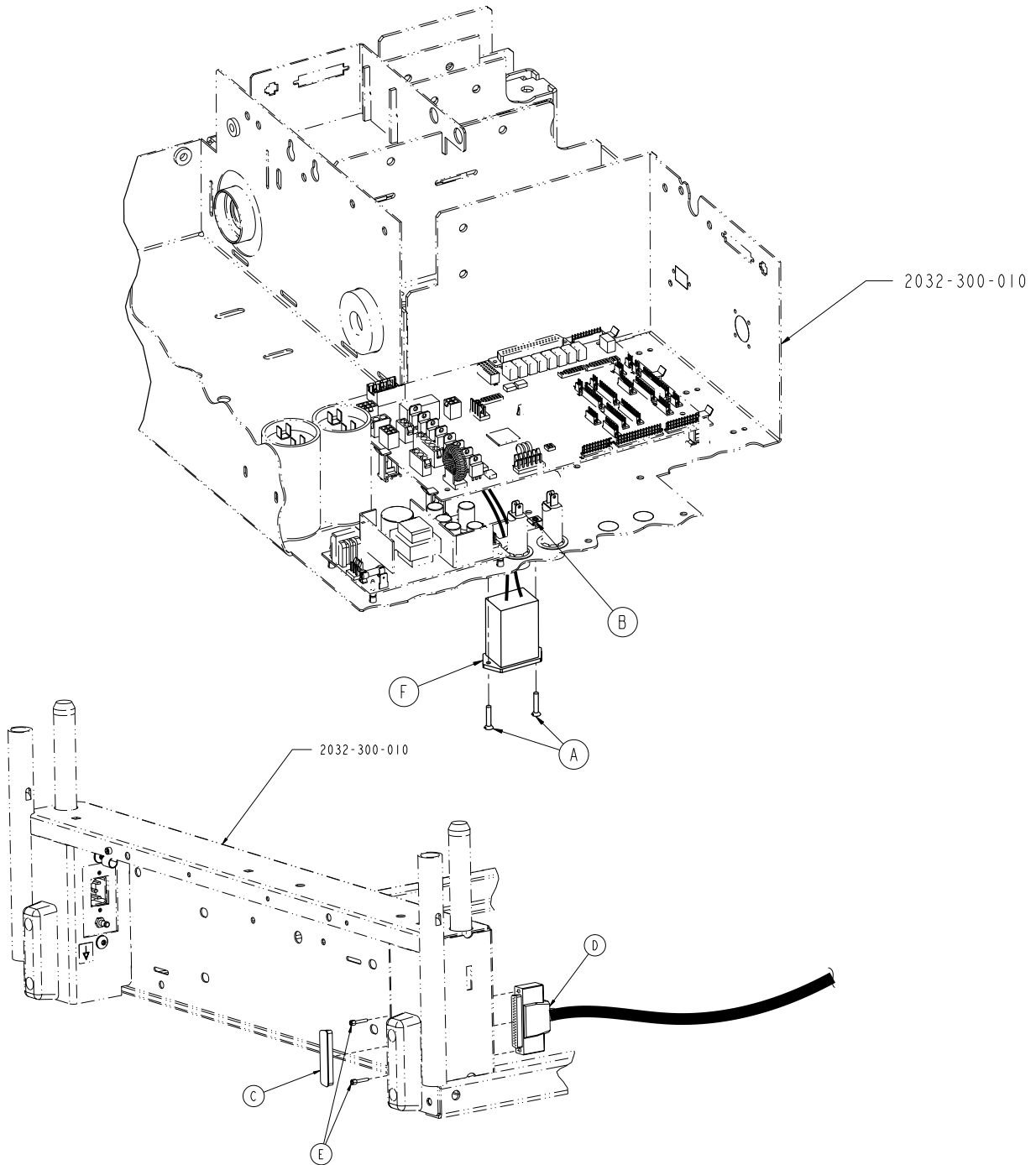


A	0001-087-000	2
B	0052-783-000	2
C	0059-710-000	1
D	2035-031-806	1
E	3001-300-007	2
F	5010-080-020	1

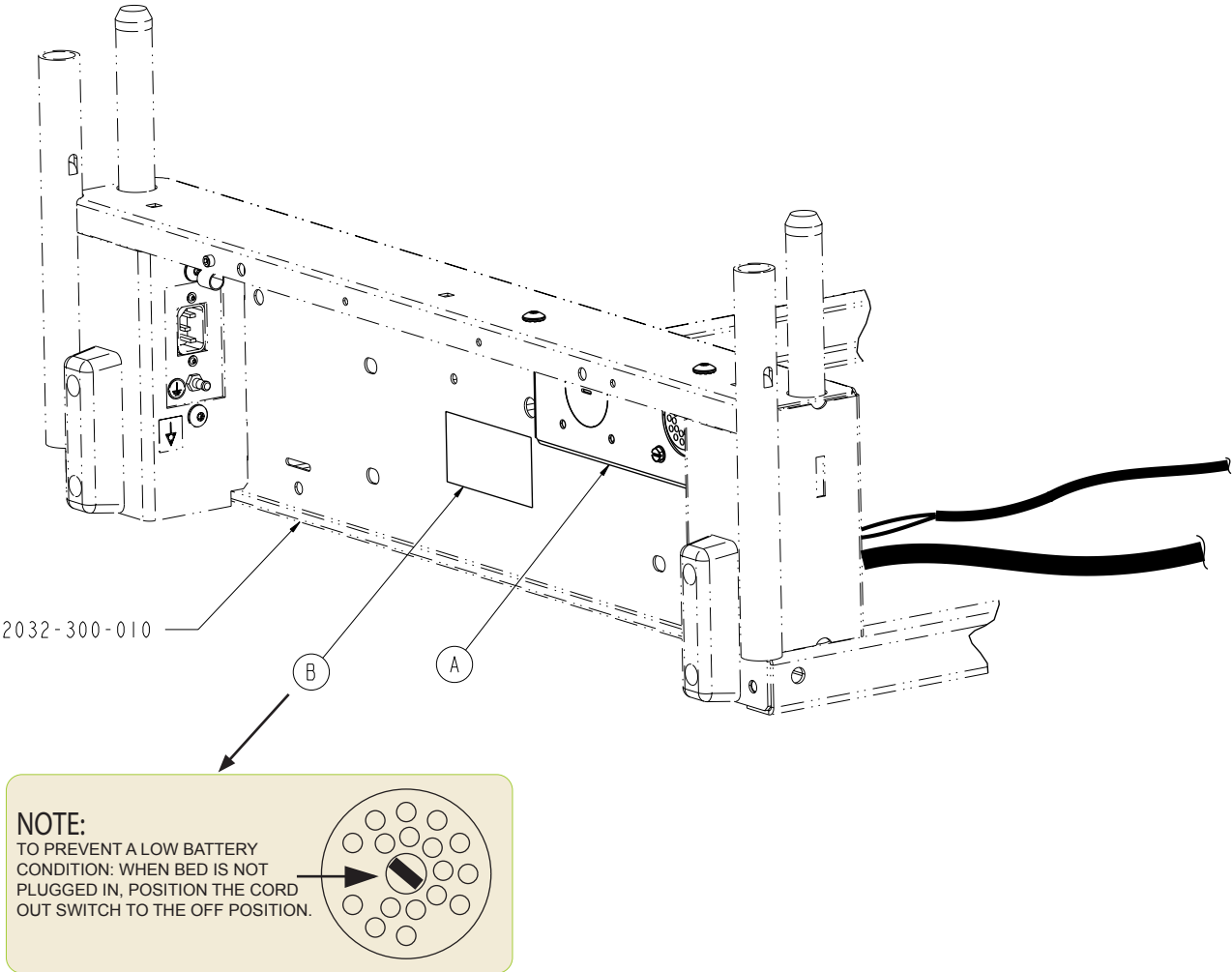


A	2031-030-204	1
B	5000-090-028	1
	2031-006-024 (Polish)	
	2031-008-024 (Portuguese)	



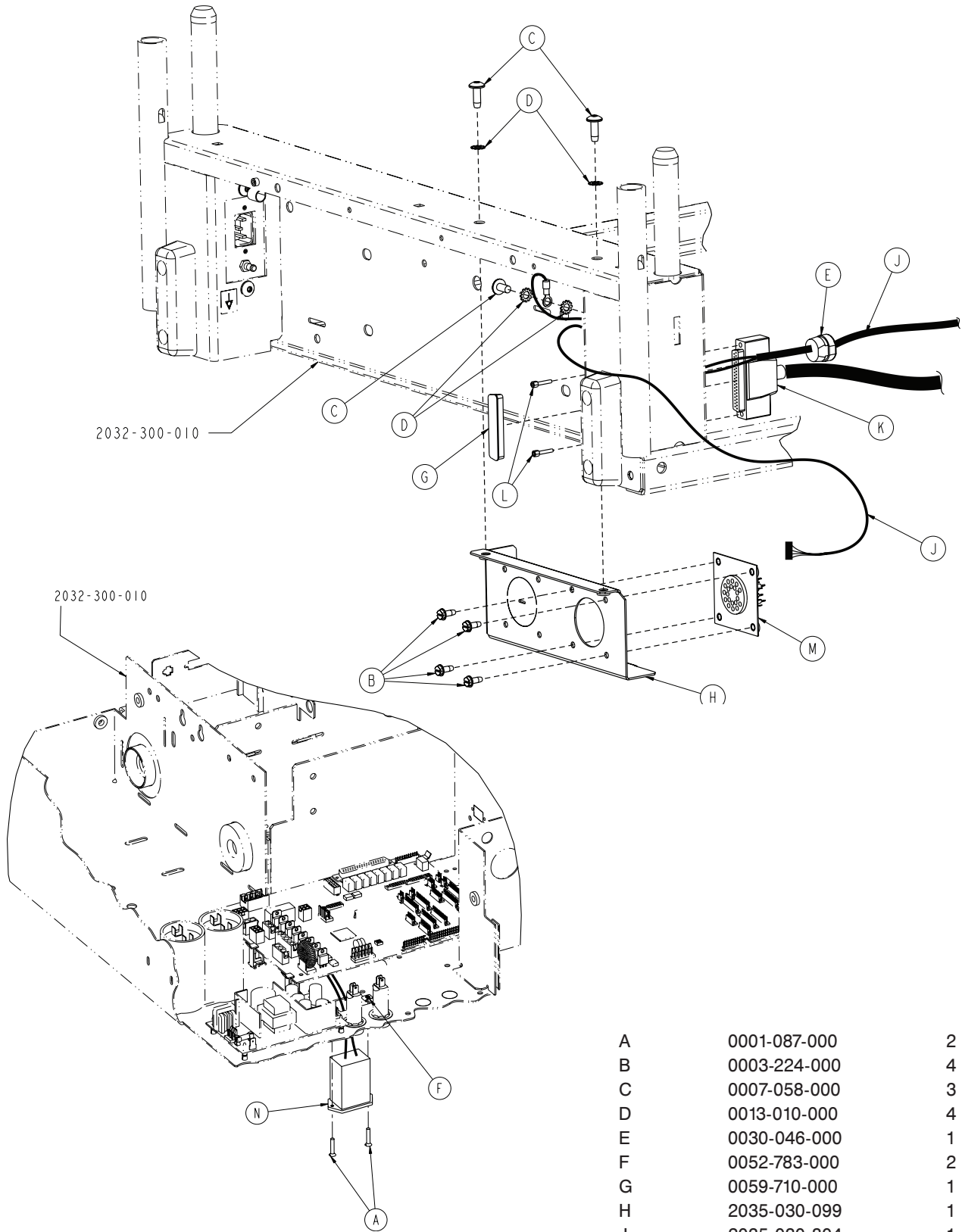





A	0001-087-000	2
B	0003-224-000	4
C	0007-058-000	3
D	0013-010-000	4
E	0030-046-000	1
F	0052-783-000	2
G	0059-710-000	1
H	2035-030-099	1
J	2035-030-804	1
K	2035-031-806	1
L	3001-300-007	2
M	3001-314-920	1
N	5010-080-020	1

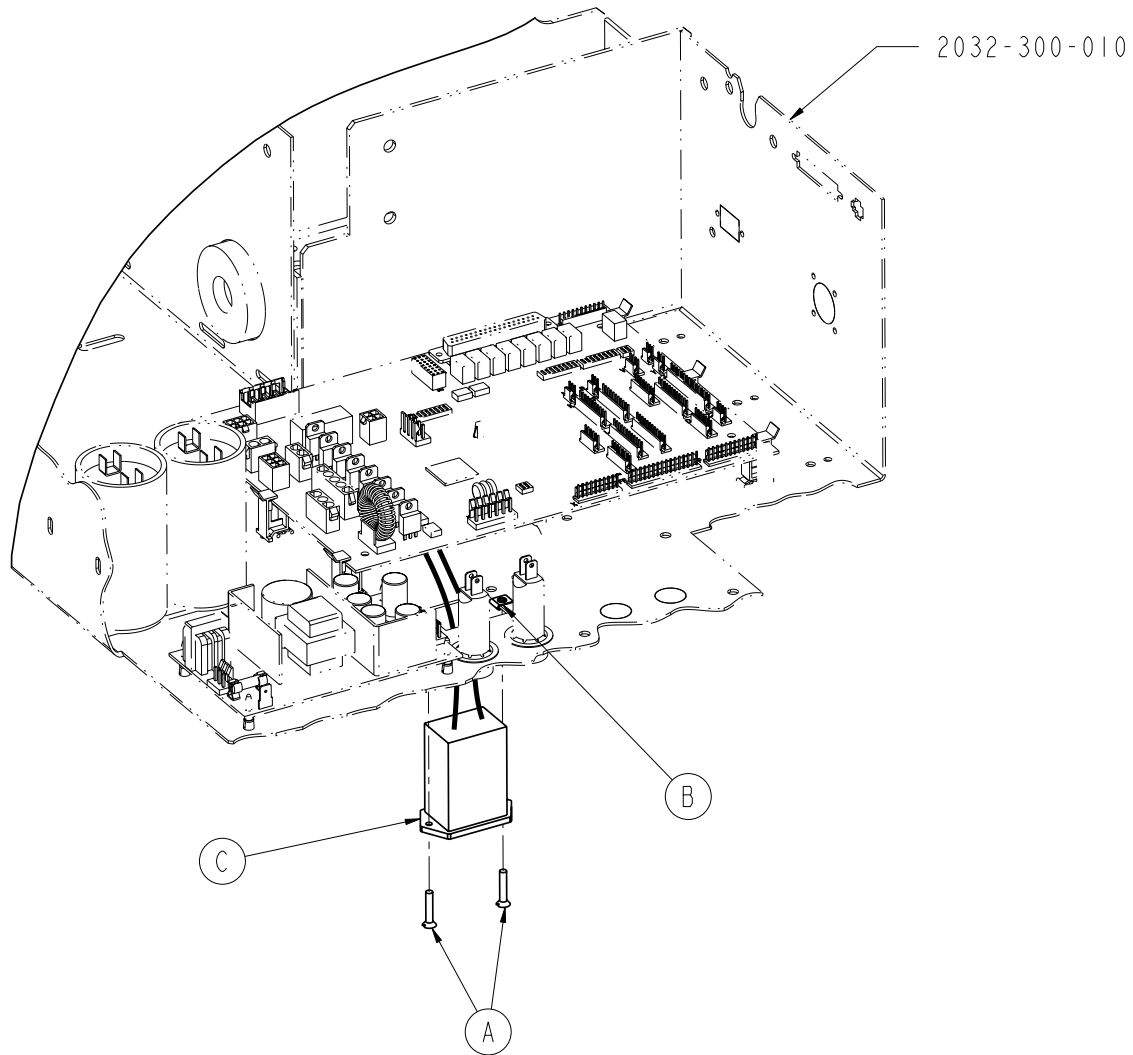



A	0001-087-000	2
B	0052-783-000	2
C	5010-080-020	1

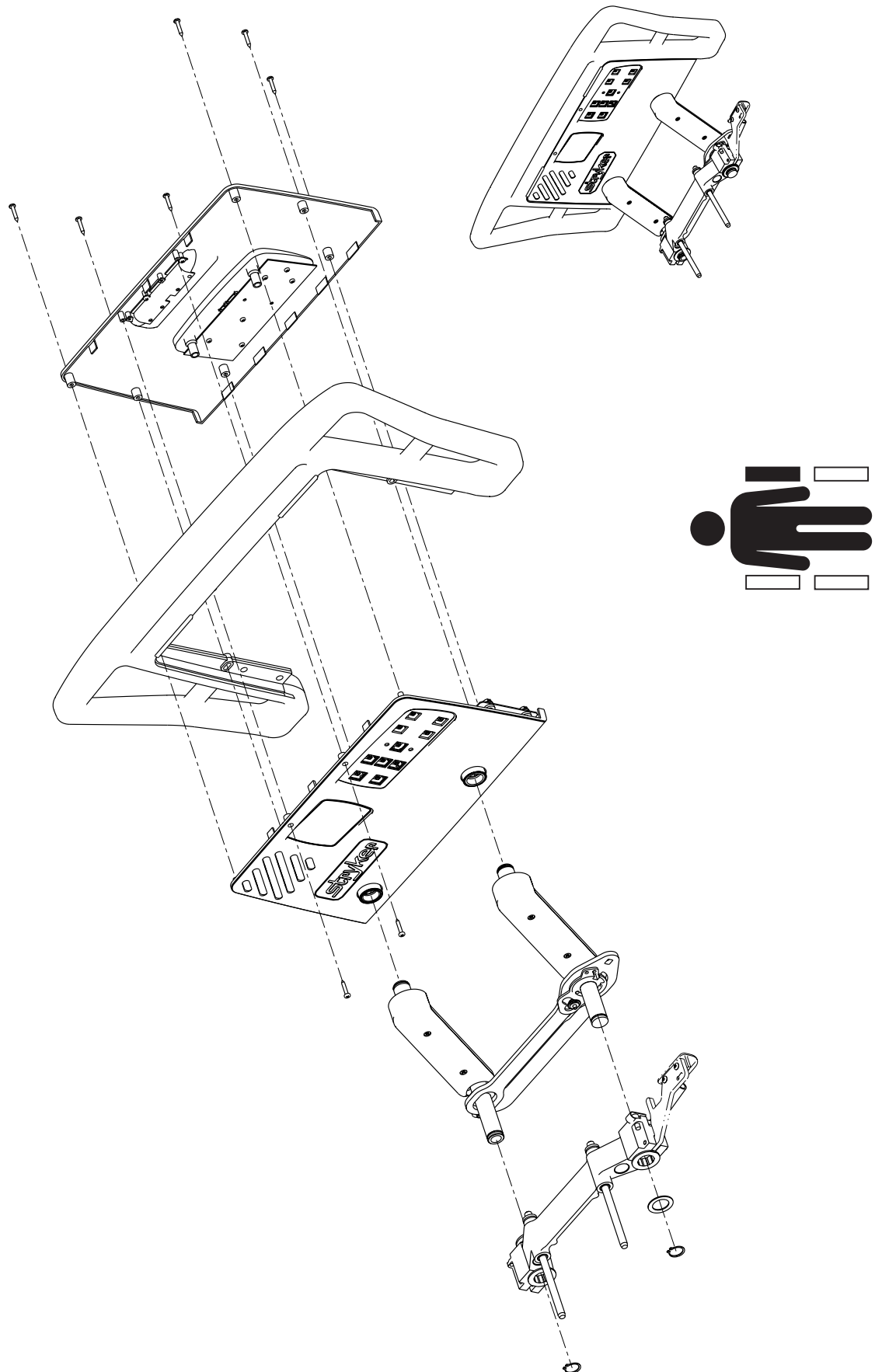

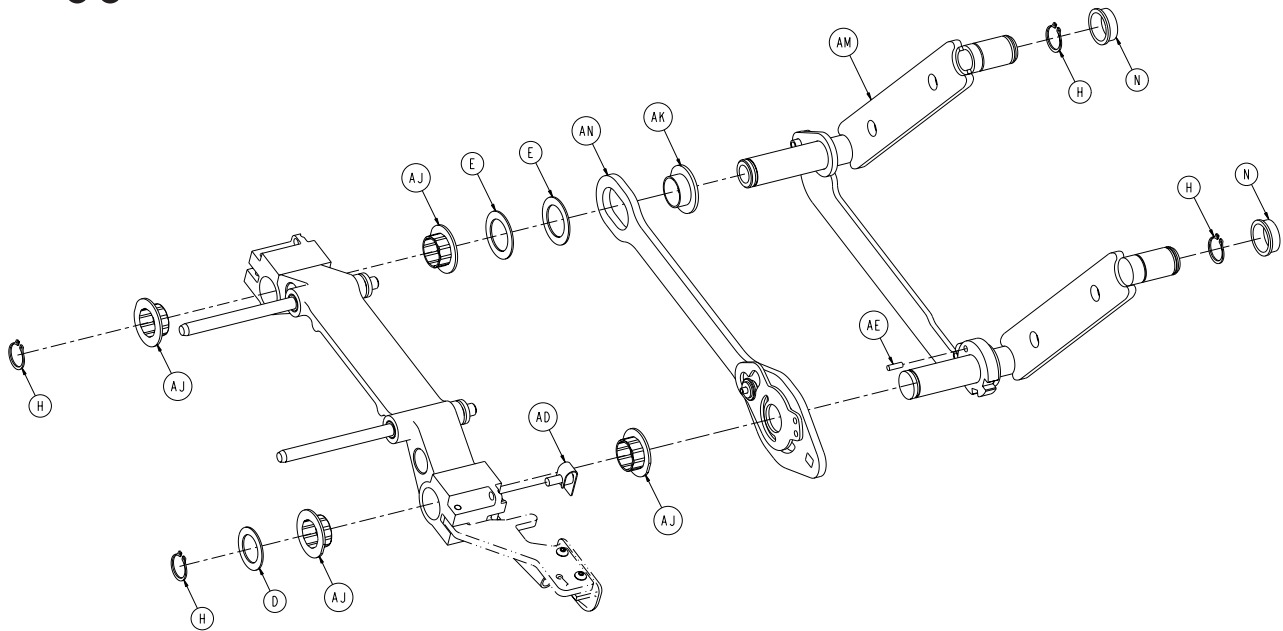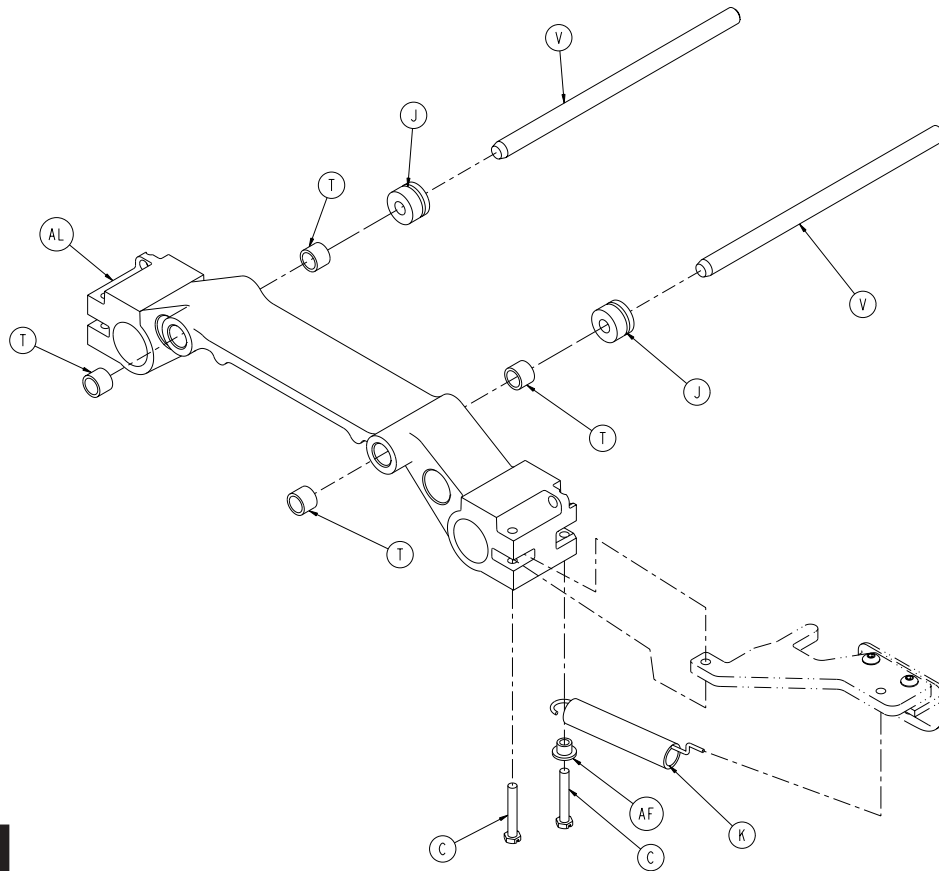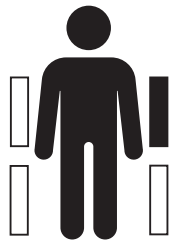

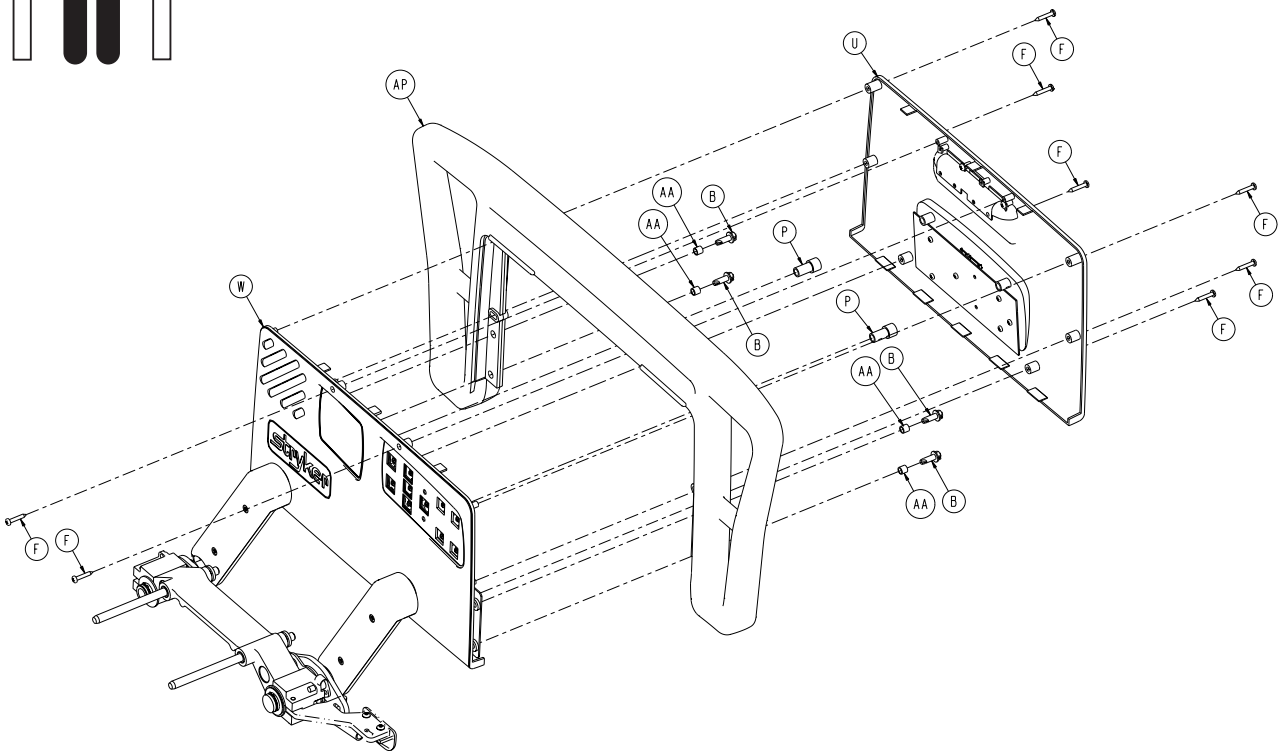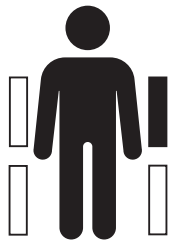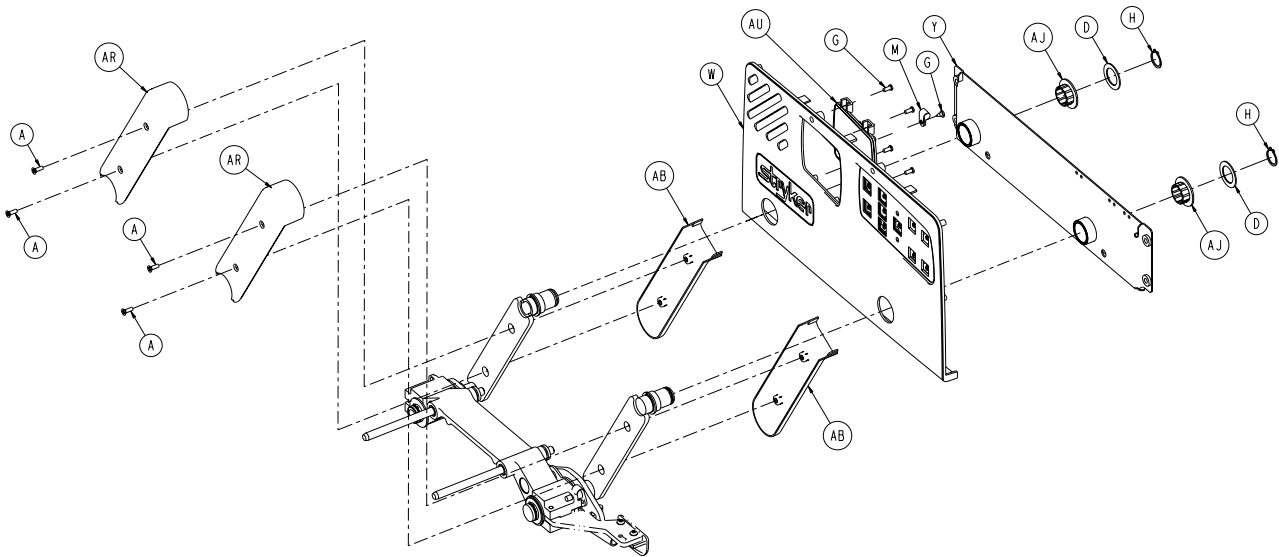


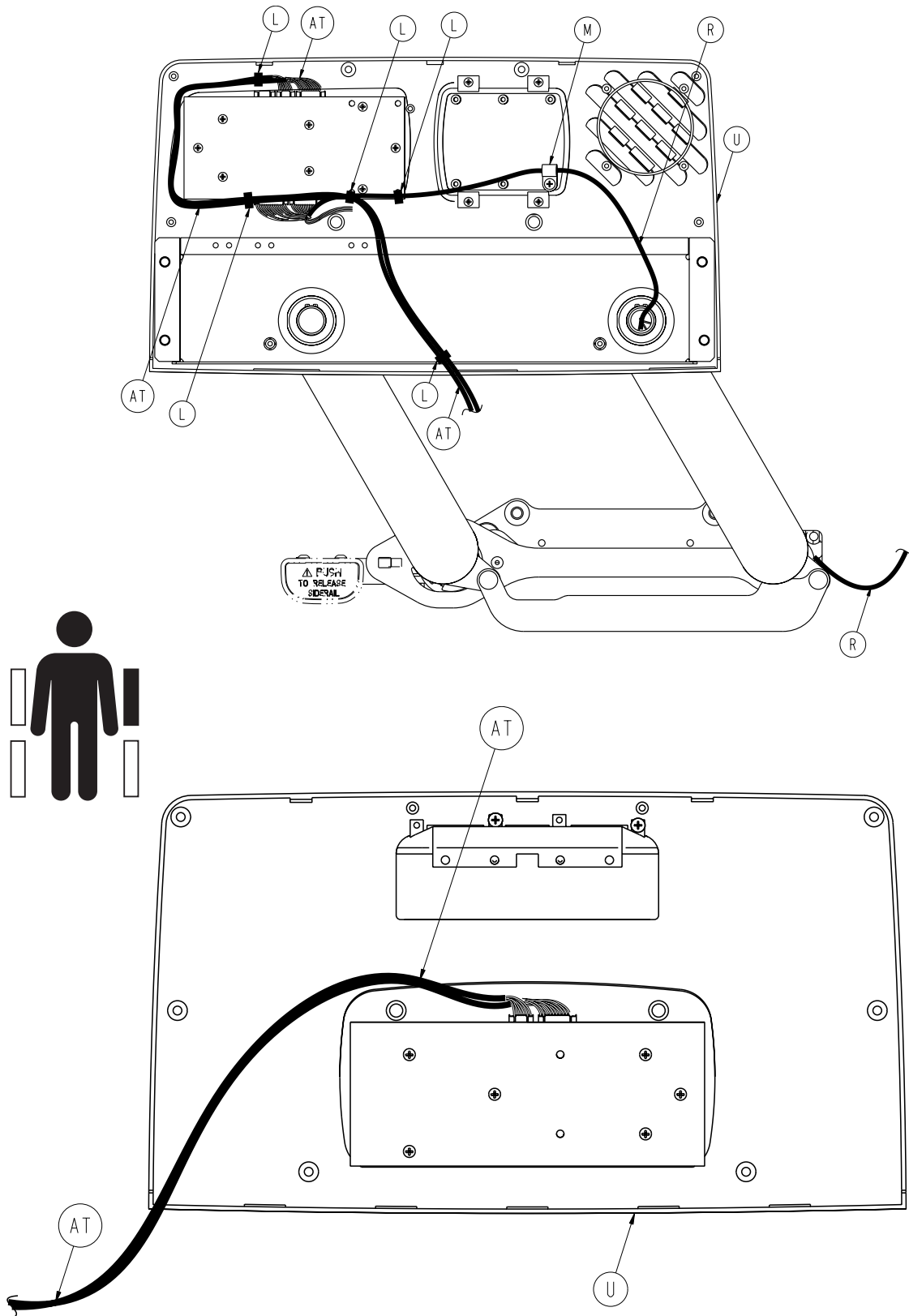


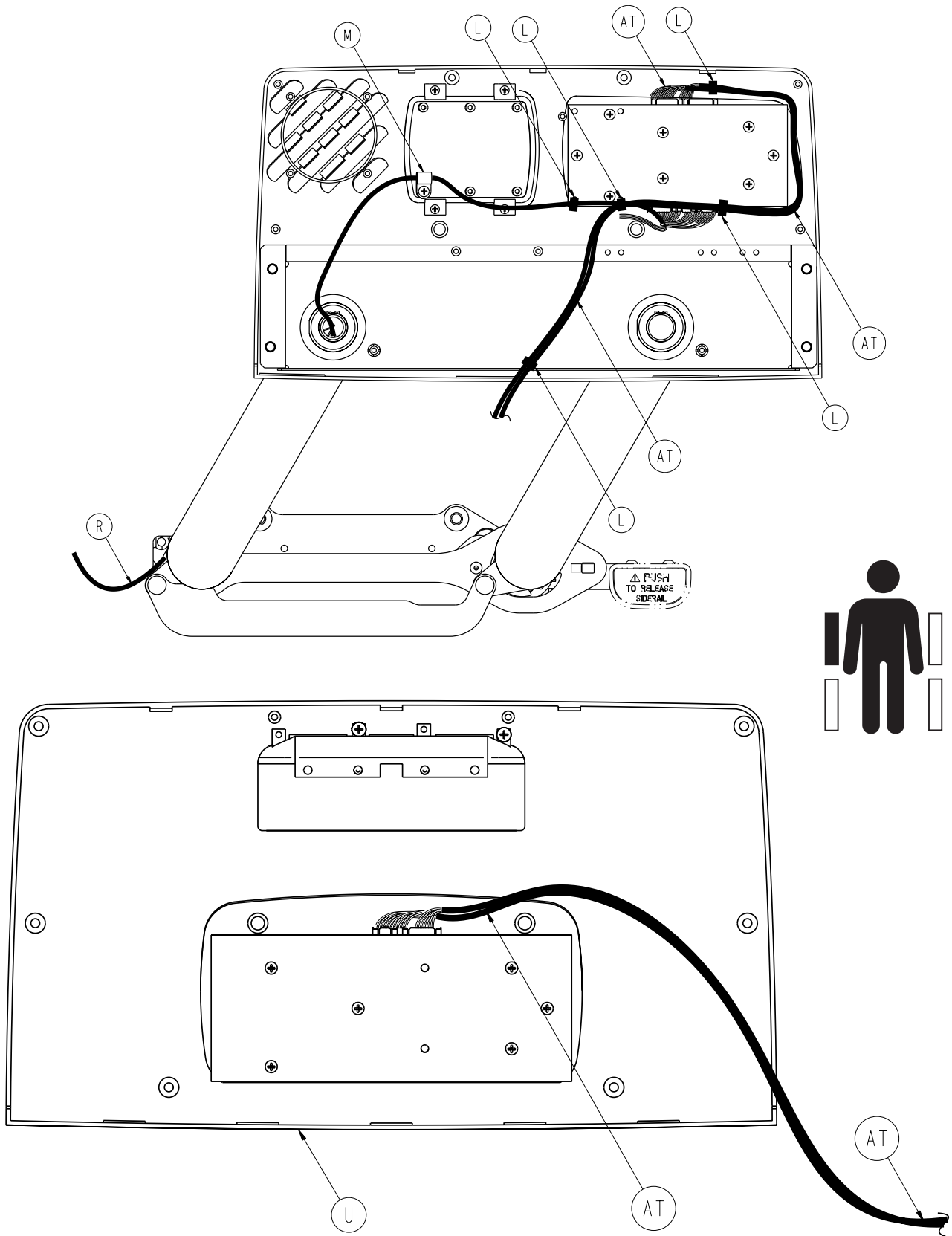



2031-400-105 / 2031-400-205

2031-400-105			2031-400-205		
A	0001-072-000	4	A	0001-072-000	4
B	0003-226-000	4	B	0003-226-000	4
C	0003-344-000	2	C	0003-344-000	2
D	0011-343-000	3	D	0011-343-000	3
E	0014-093-000	2	E	0014-093-000	2
F	0023-090-000	8	F	0023-090-000	8
G	0023-088-000	5	G	0023-088-000	5
H	0028-128-000	6	H	0028-128-000	6
J	0030-040-000	2	J	0030-040-000	2
K	3000-200-334	1	K	3000-200-334	1
L	3000-300-114	5	L	3000-300-114	5
M	3000-300-478	1	M	3000-300-478	1
N	3000-400-513	2	N	3000-400-513	2
P	3000-400-523	2	P	3000-400-523	2
R	2035-020-802	1	R	2035-020-802	1
T	3000-400-557	4	T	3000-400-557	4
U	2032-400-050	1	U	2032-400-050	1
V	2035-400-570	2	V	2035-400-570	2
W	3001-400-040	1	W	3001-400-045	1
Y	3003-400-130	1	Y	3003-400-230	1
AA	3001-400-558	4	AA	3001-400-558	4
AB	3001-400-619	2	AB	3001-400-619	2
AD	3002-400-090	1	AD	3002-400-090	1
AE	3002-400-505	1	AE	3002-400-505	1
AF	3002-400-509	1	AF	3002-400-509	1
AH	3002-400-512	2	AH	3002-400-512	2
AJ	3002-400-513	6	AJ	3002-400-513	6
AK	3002-400-519	1	AK	3002-400-519	1
AL	3002-400-528	1	AL	3002-400-528	1
AM	3003-400-005	1	AM	3003-400-010	1
AN	3003-400-070	1	AN	3003-400-075	1
AP	3003-400-515	1	AP	3003-400-515	1
AR	5000-020-005	2	AR	5000-020-005	2
AT	2035-020-804	1	AT	2035-020-803	1
AU	3001-400-535	1	AU	3001-400-535	1

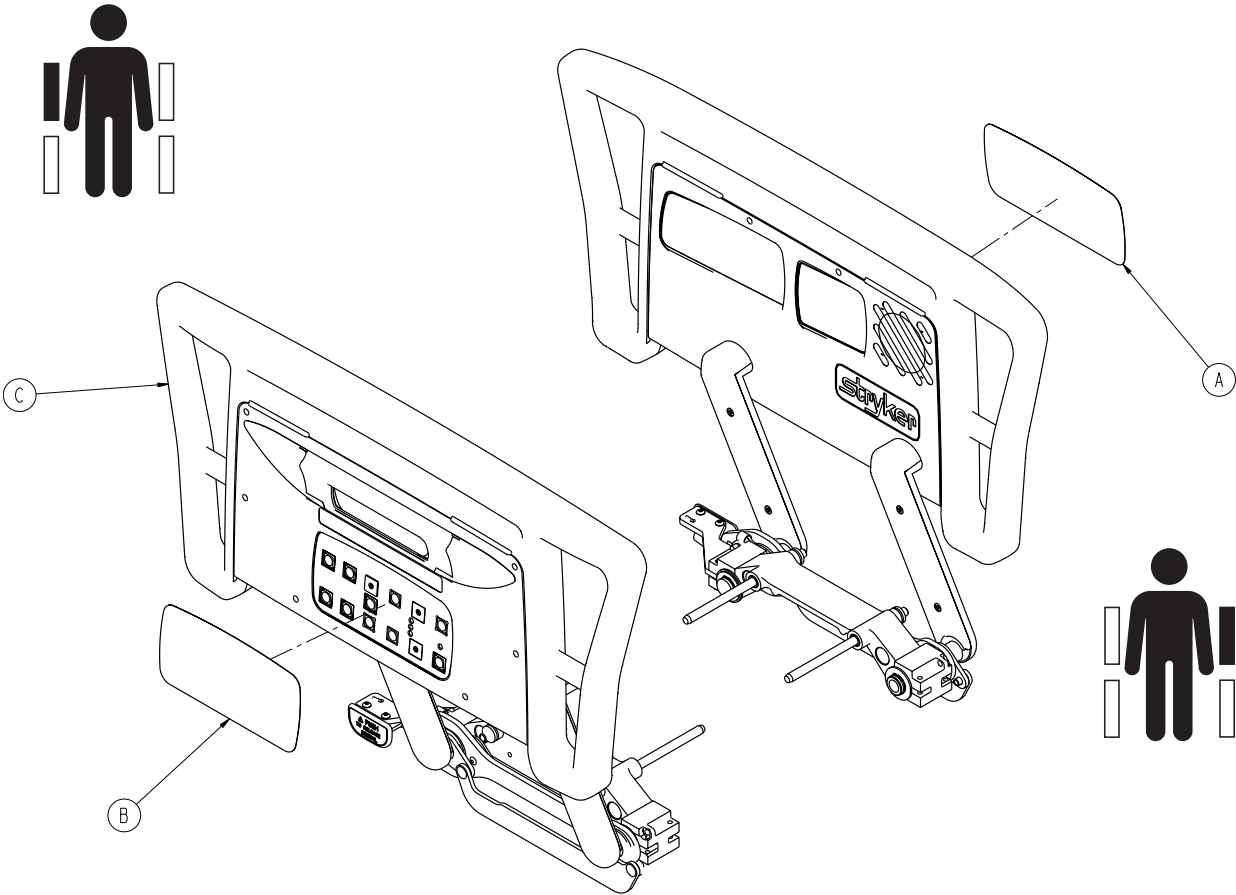


A	2030-000-300 (English)	1
	2031-006-022 (Polish)	
	2031-008-022 (Portuguese)	
B	2030-000-400 (English)	1
	2031-006-023 (Polish)	
	2031-008-023 (Portuguese)	
C	2031-999-160	1

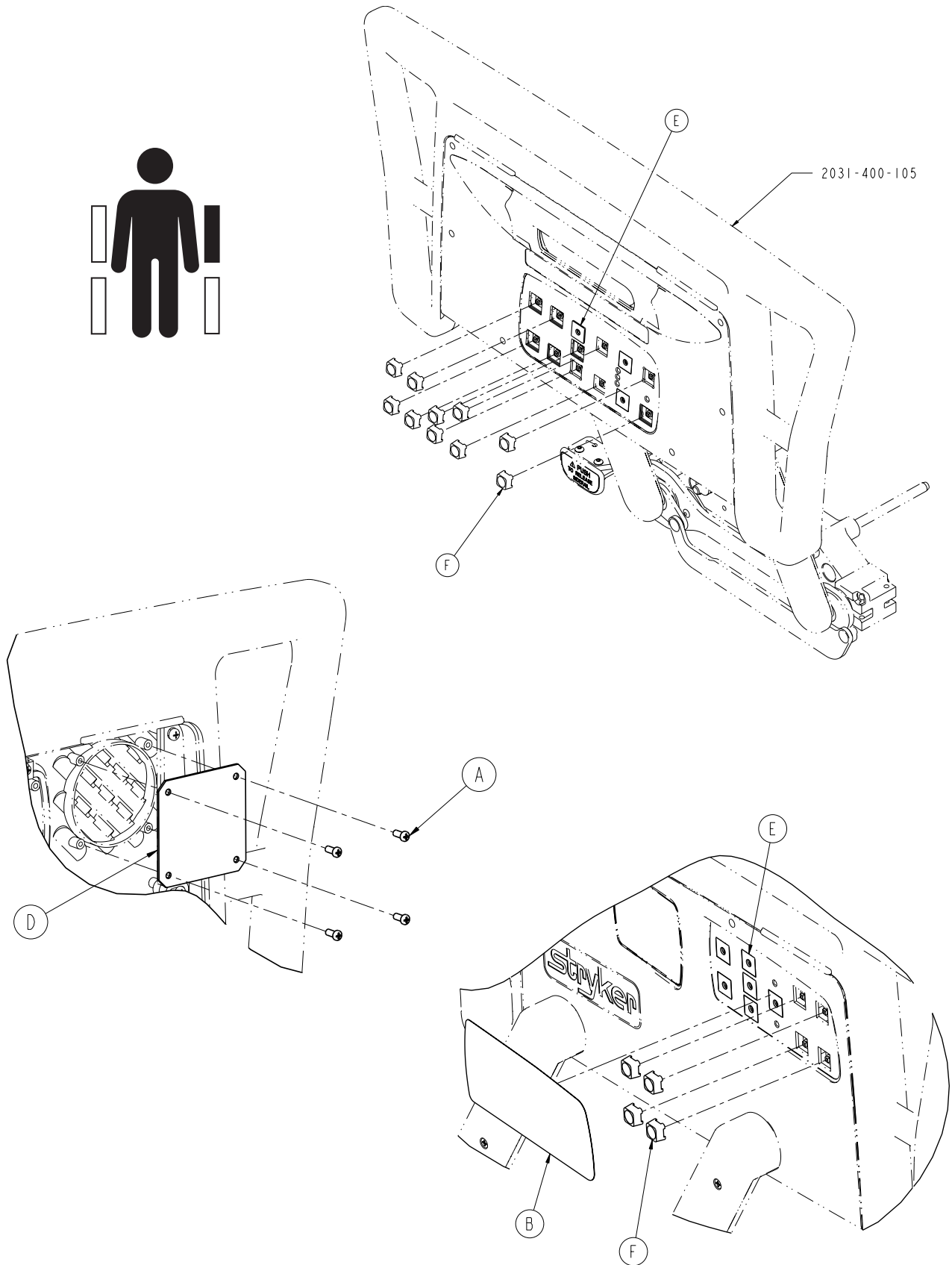
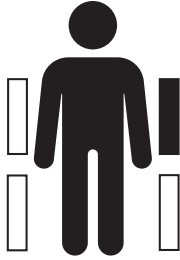



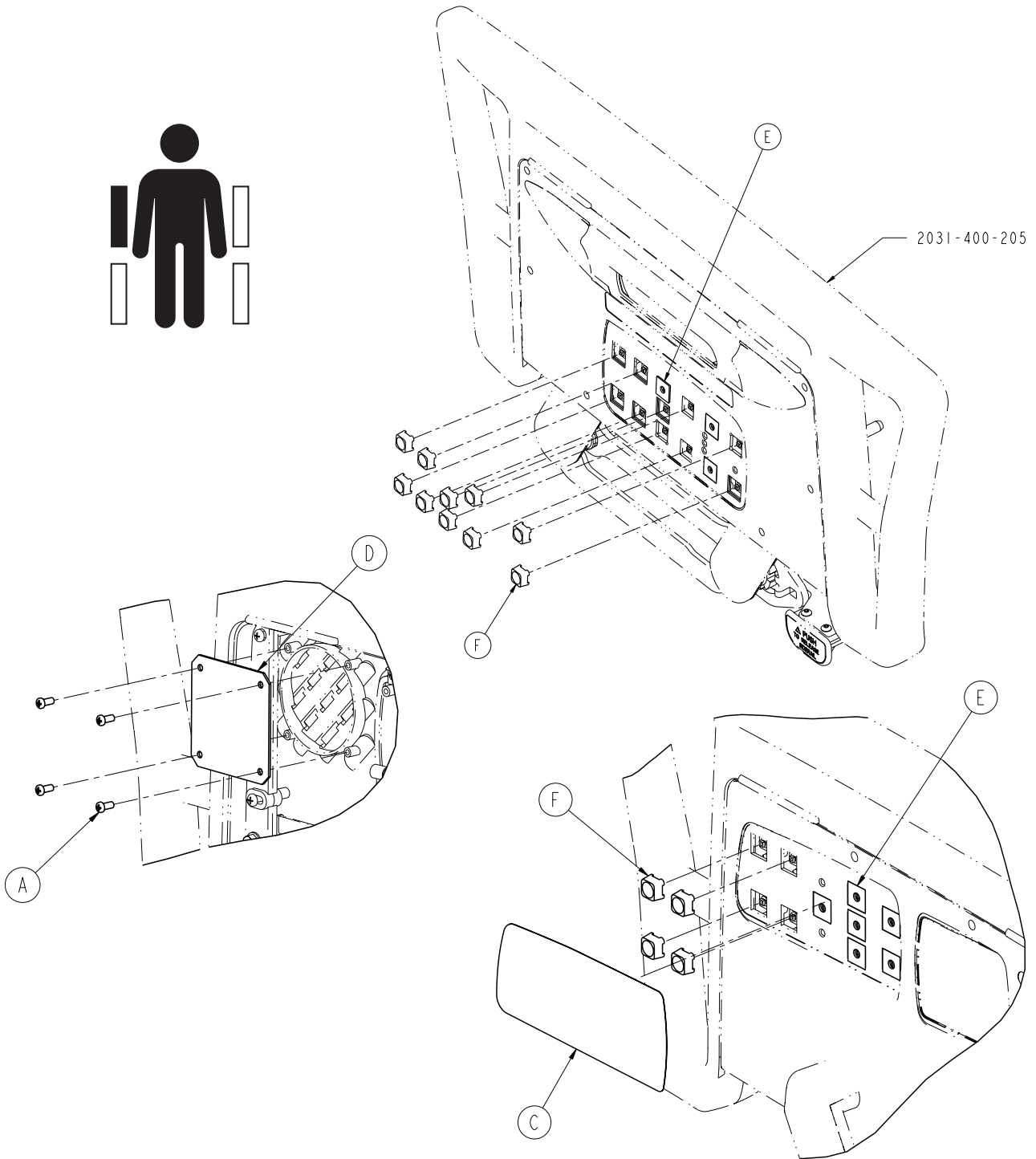


A	0023-088-000	8
B	2035-000-100	1
C	2035-000-200	1
D	3001-400-517	2
E	3001-400-522	18
F	3001-400-953	28


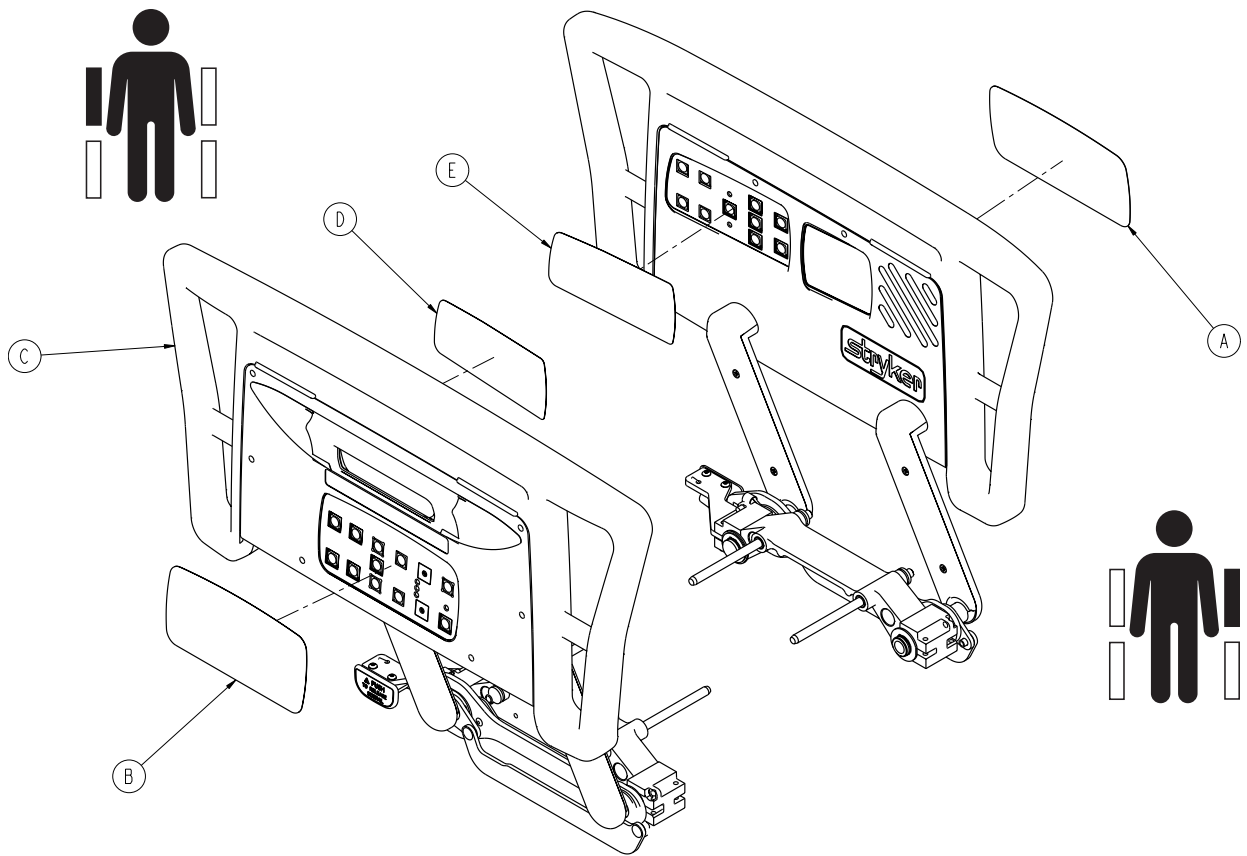


A	2030-000-301 (English)	1
	2031-006-018 (Polish)	
	2031-008-018 (Portuguese)	
B	2030-000-401 (English)	1
	2031-006-019 (Polish)	
	2031-008-019 (Portuguese)	
C	2031-999-170	1
D	2035-000-102 (English)	1
	2031-006-020 (Polish)	
	2031-008-020 (Portuguese)	
E	2035-000-202 (English)	1
	2031-006-021 (Polish)	
	2031-008-021 (Portuguese)	

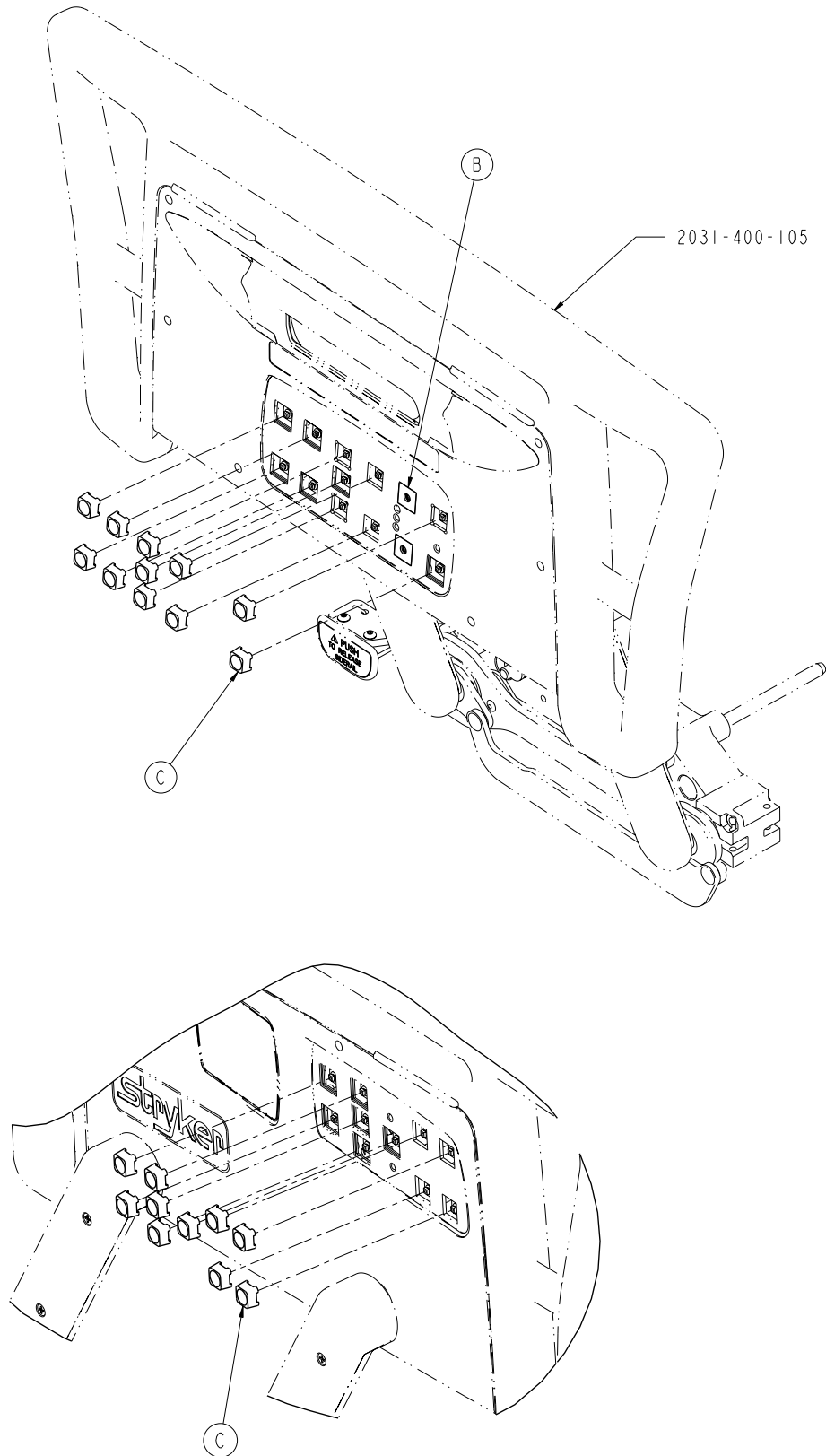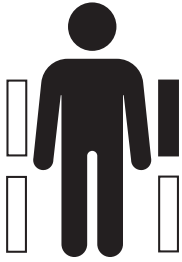

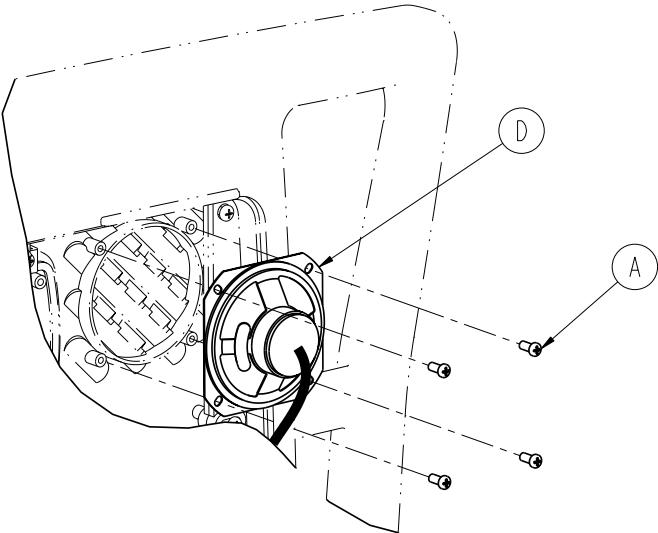
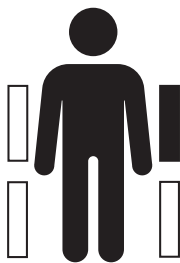


A	0023-088-000	8
B	3001-400-522	4
C	3001-400-953	42
D	3001-403-831	2

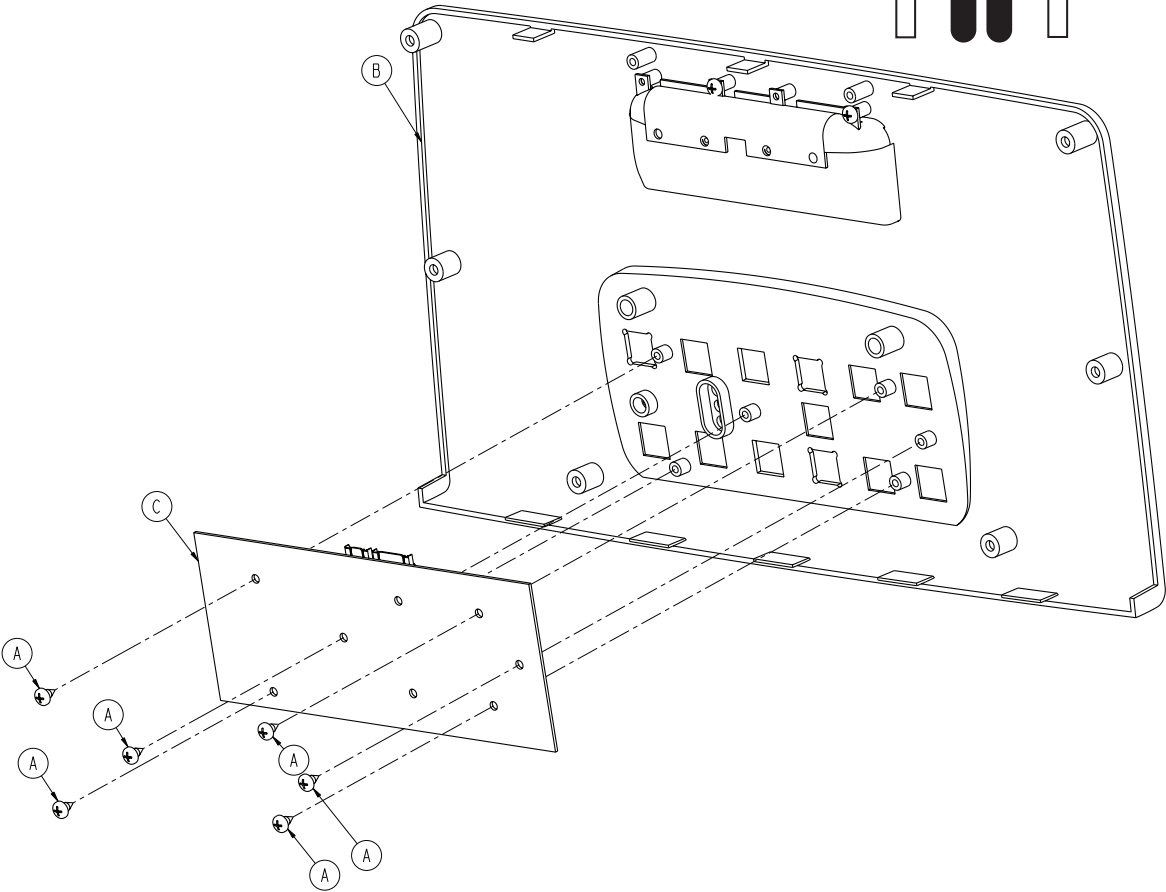
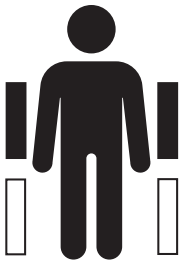


A	0023-088-000	6
B	2035-400-050	1
C	2035-400-900	1



3001-400-045 / 3001-400-040

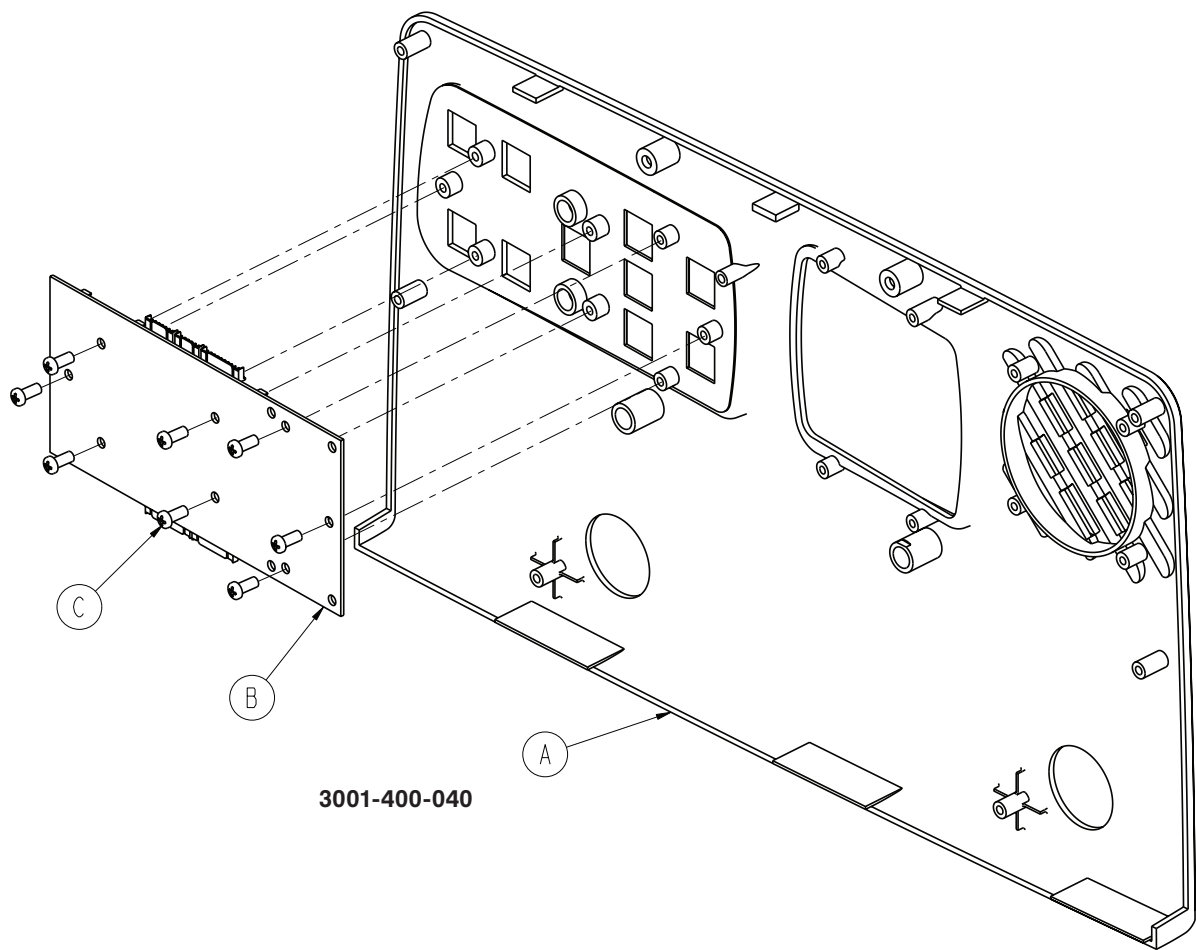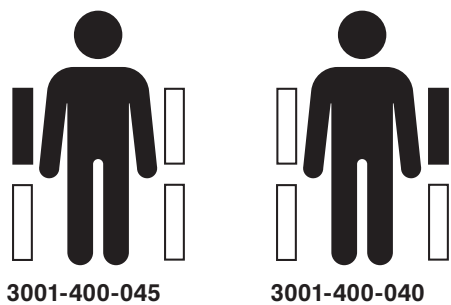


A	3001-400-101	1
	3001-400-201	1
B	3001-400-930	1
C	0023-088-000	8

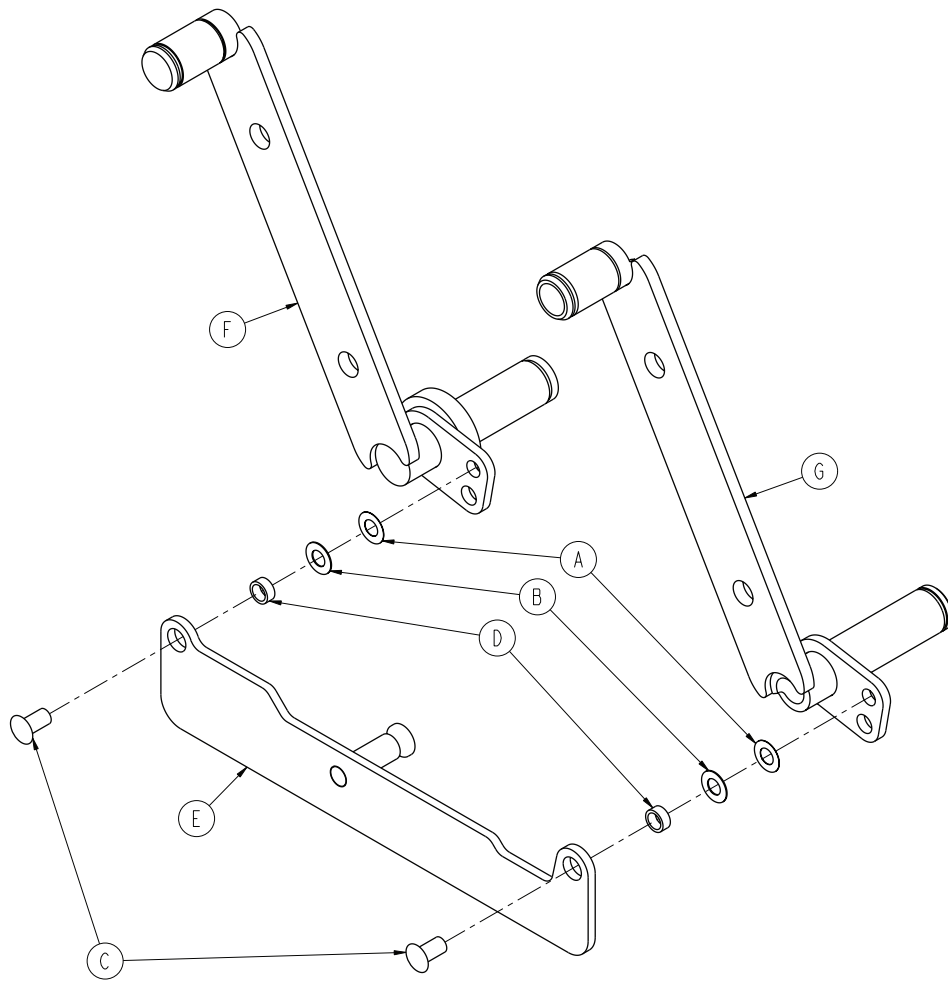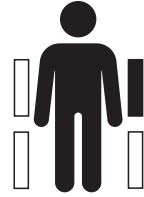



A	0011-377-000	2
B	0011-491-000	2
C	3001-400-501	2
D	3003-401-001	2
E	3003-402-112	1
F	3003-402-127	1
G	3003-402-128	1

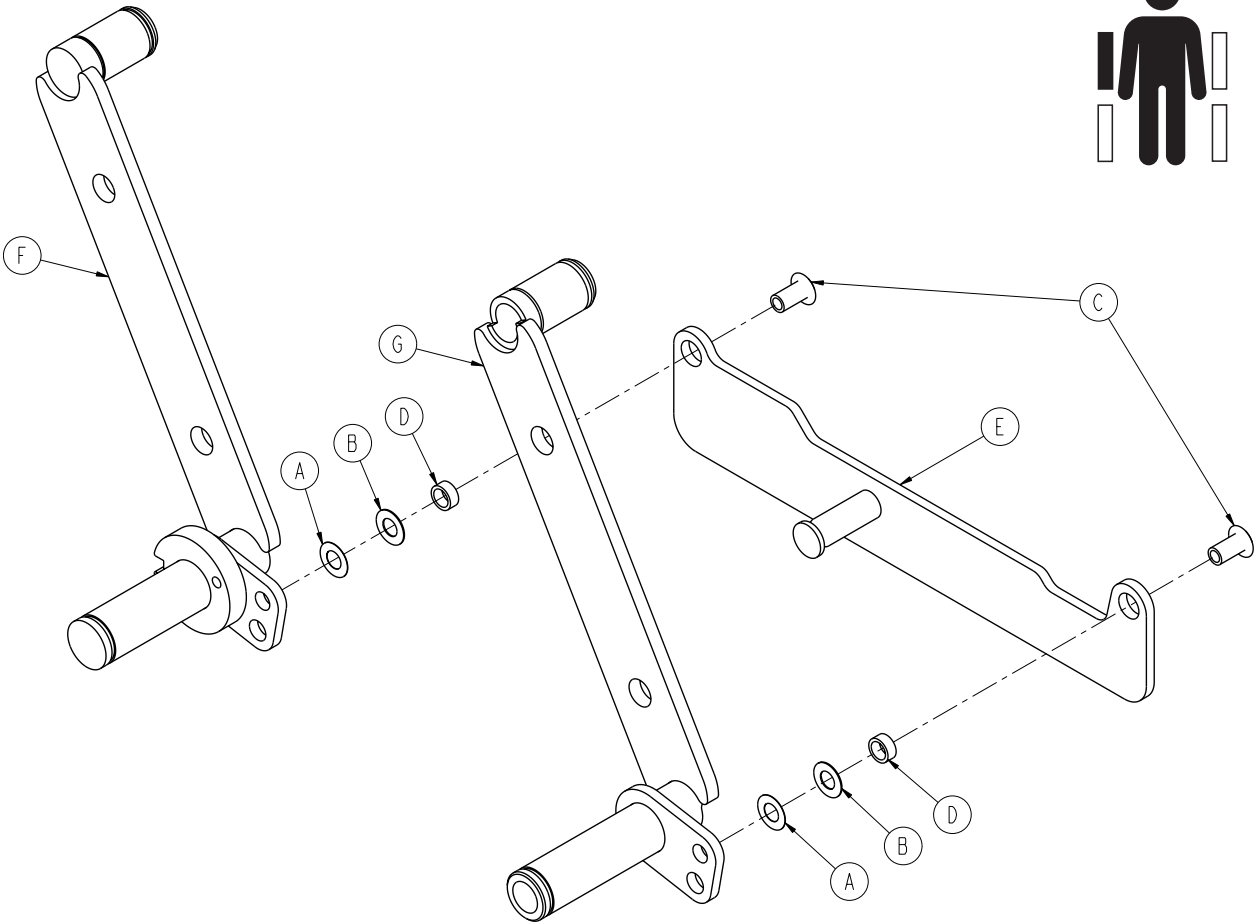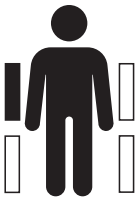



A	0011-377-000	2
B	0011-491-000	2
C	3001-400-501	2
D	3003-401-001	2
E	3003-402-117	1
F	3003-402-227	1
G	3003-402-228	1

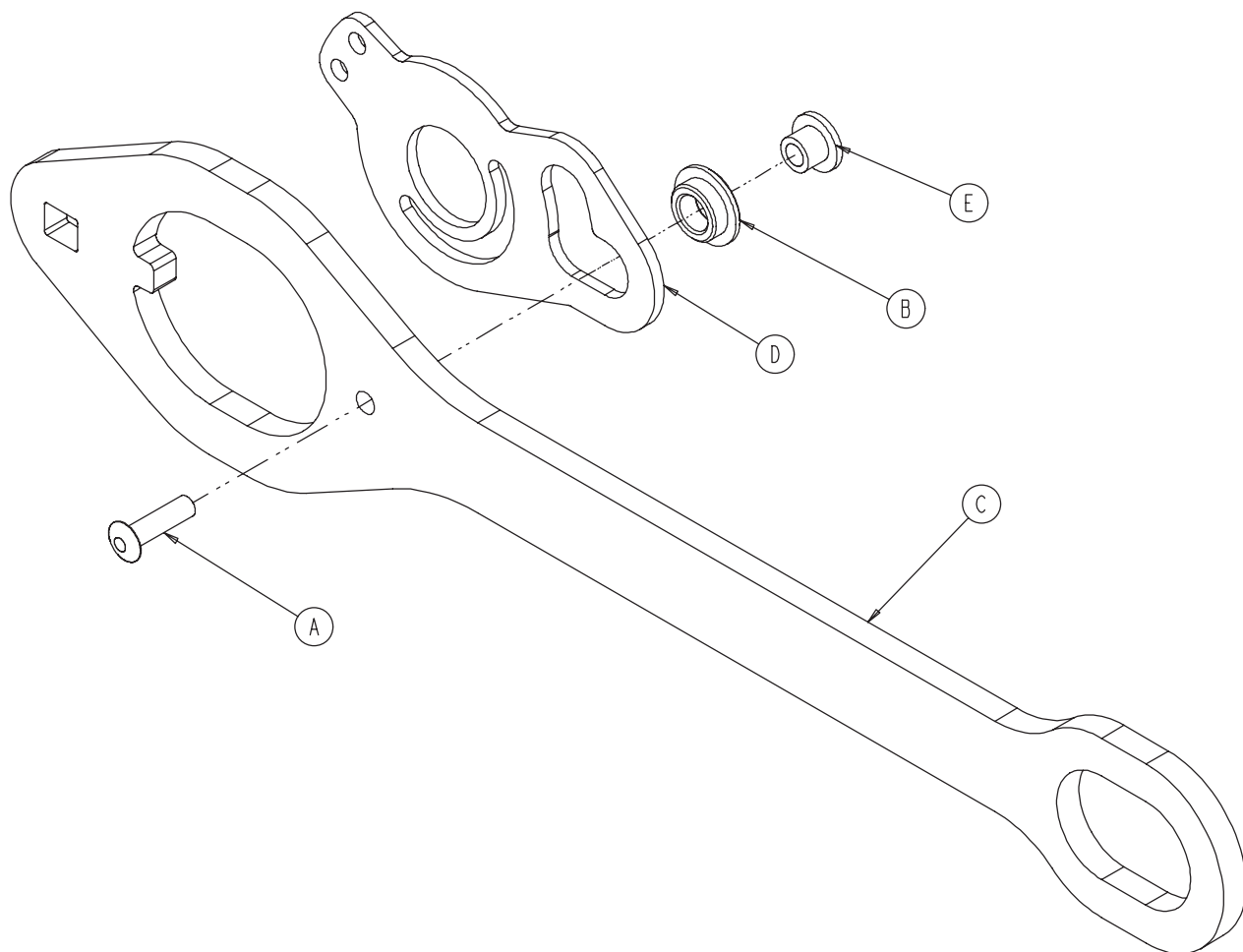



A	0025-086-000	1
B	0081-317-000	1
C	3002-400-501	1
D	3002-400-509	1
E	3002-400-503	1

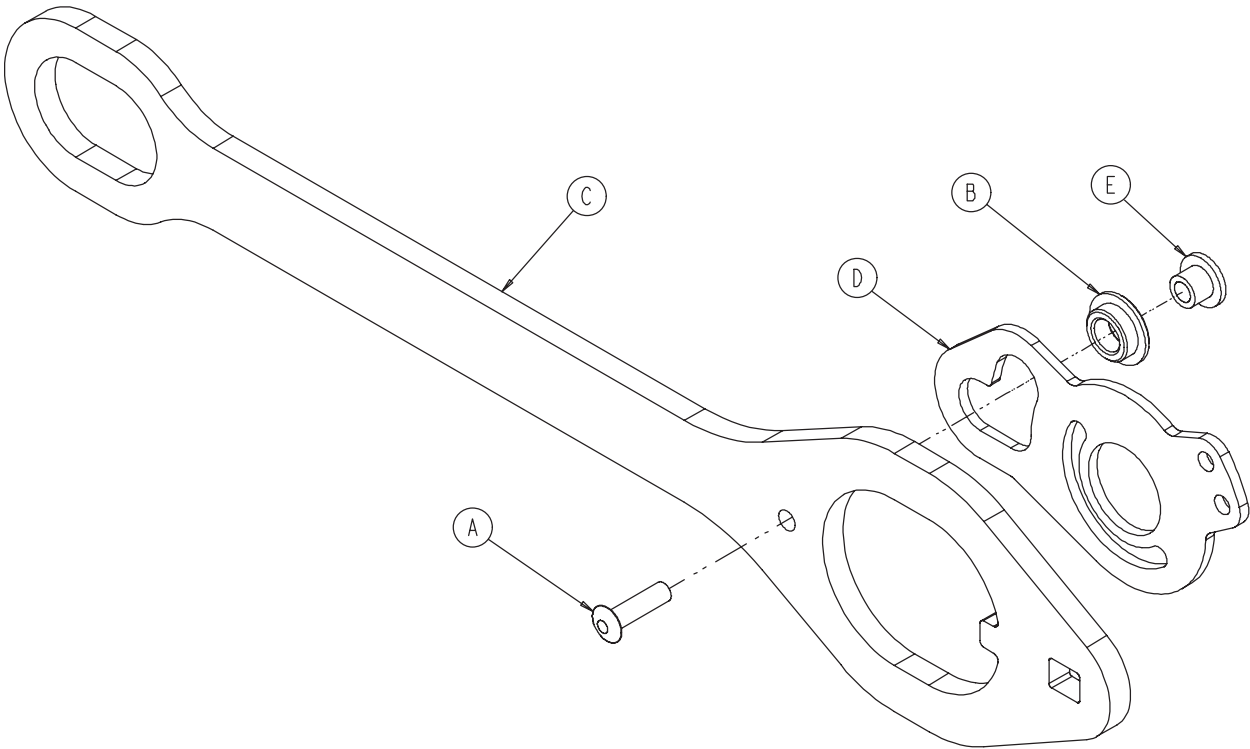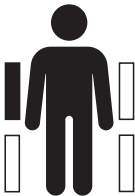


A	0025-086-000	1
B	0081-317-000	1
C	3002-400-501	1
D	3002-400-509	1
E	3003-400-503	1

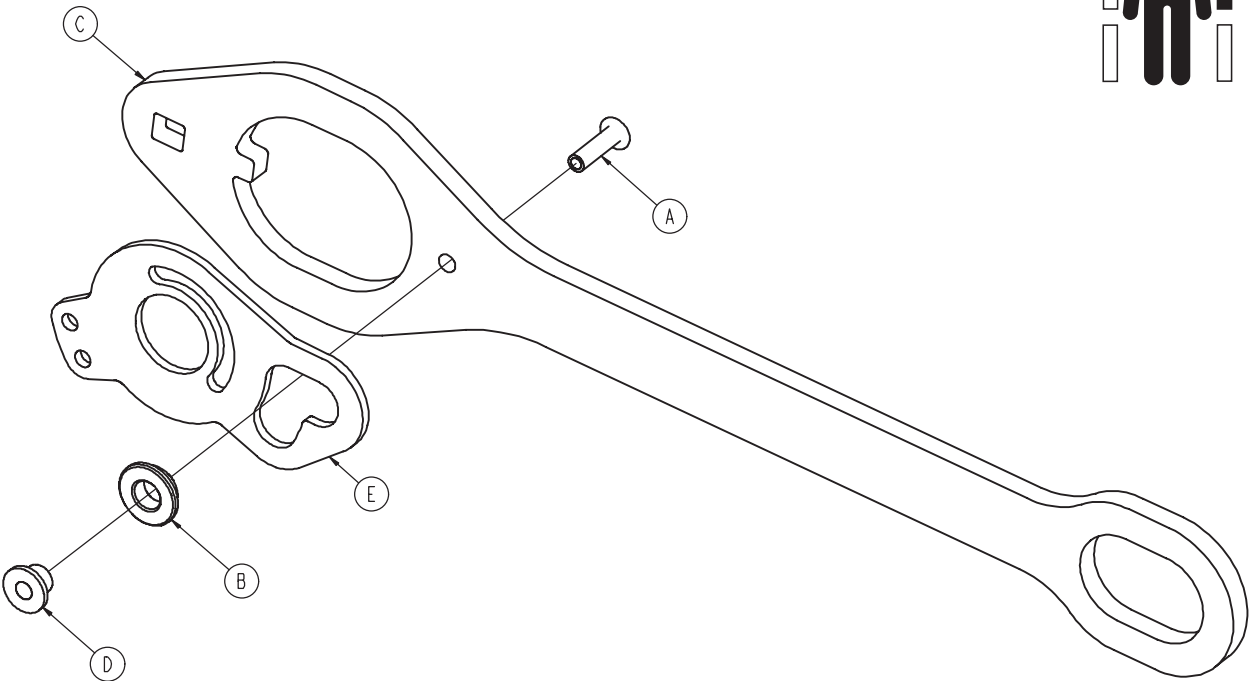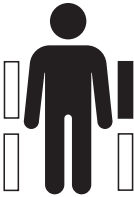


A	0025-086-000	1
B	0081-317-000	1
C	3002-400-501	1
D	3002-400-509	1
E	3003-400-503	1

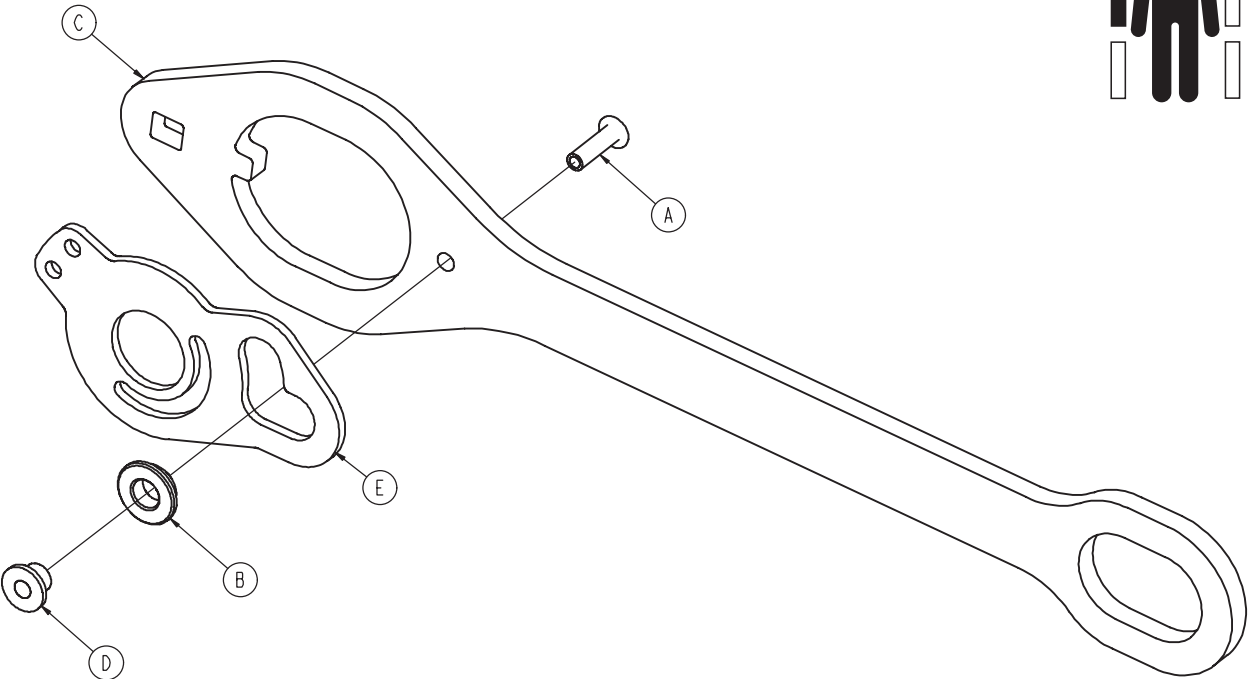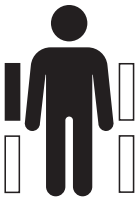


A	0025-086-000	1
B	0081-317-000	1
C	3002-400-501	1
D	3002-400-509	1
E	3003-400-503	1

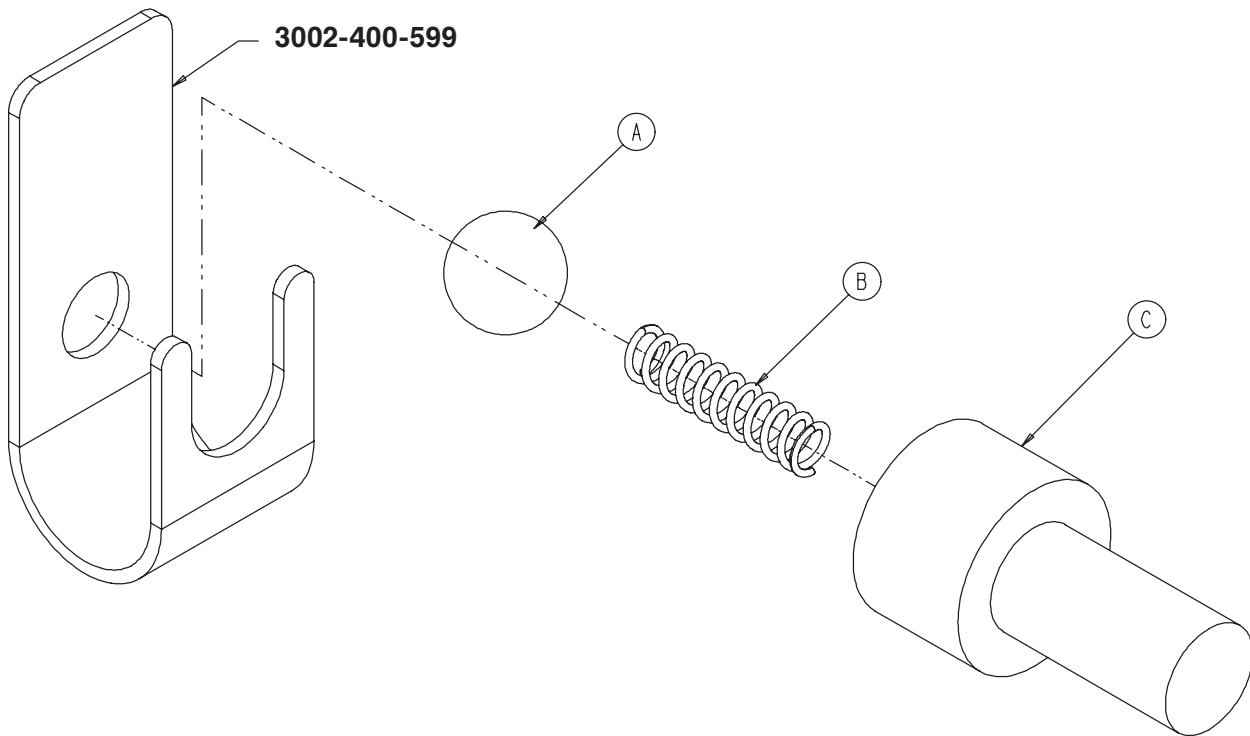


A	0031-137-000	1
B	0038-464-000	1
C	3002-400-524	1


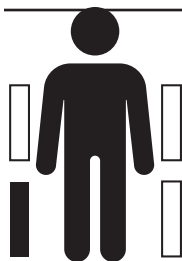





2031-400-305

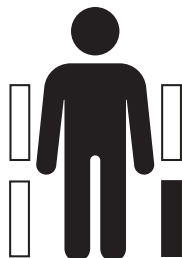


2031-400-405

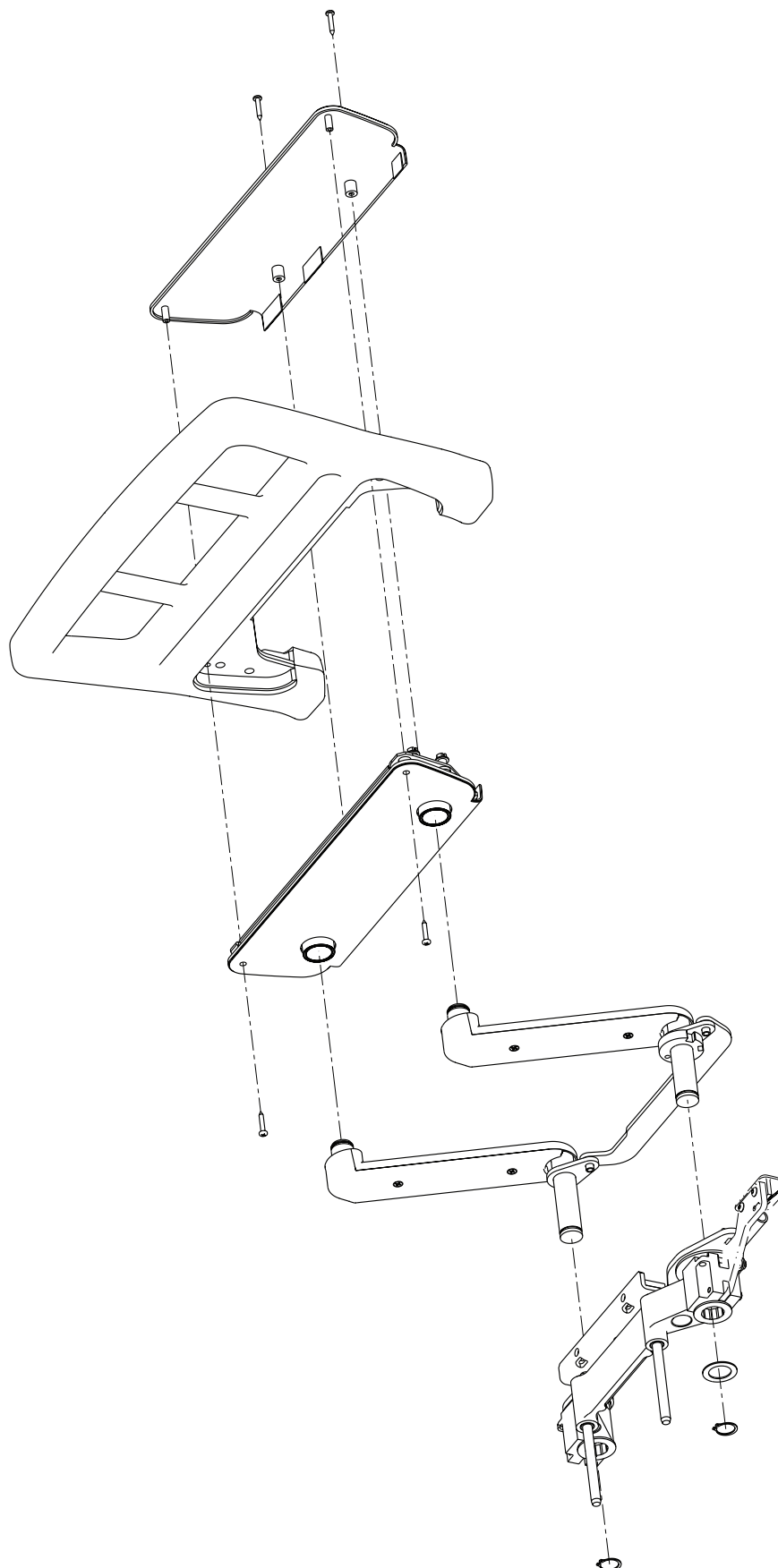

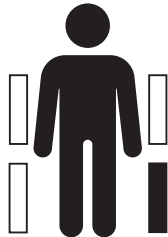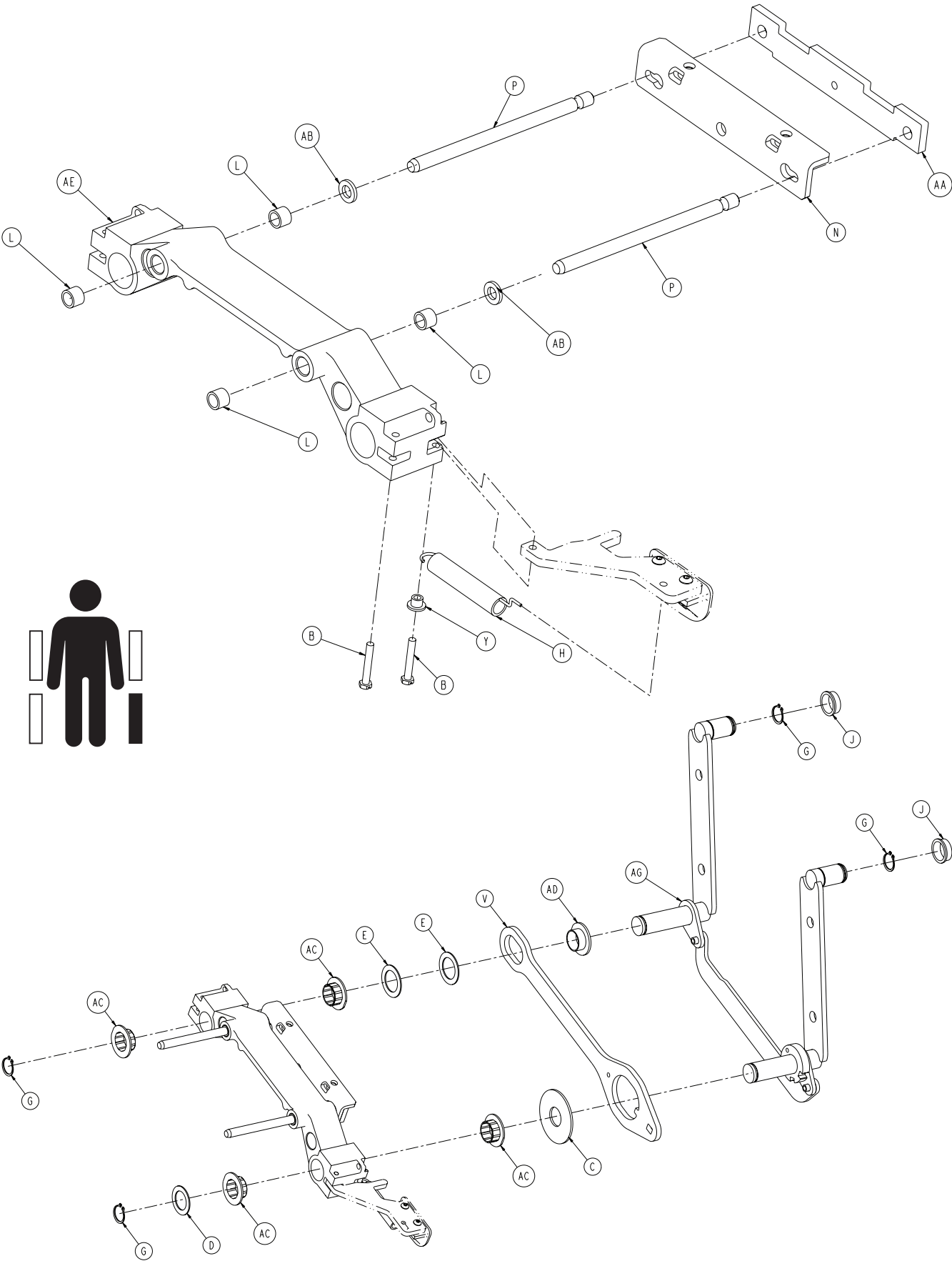


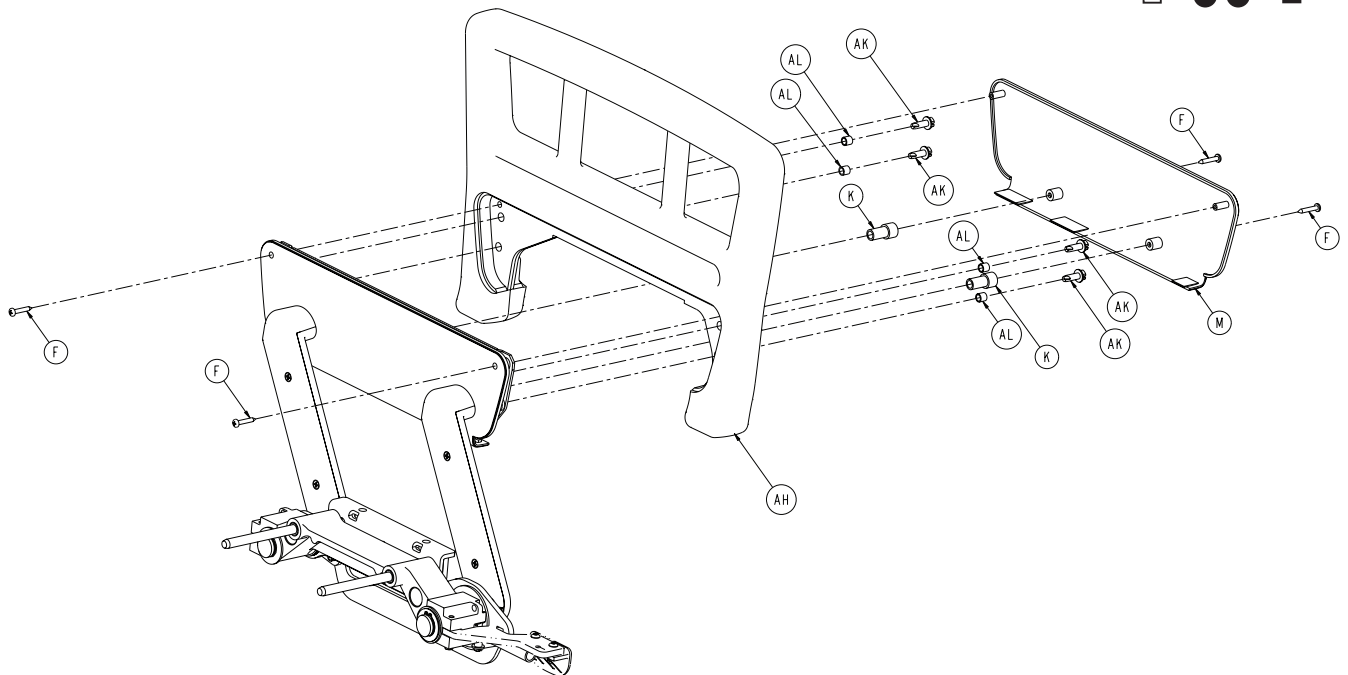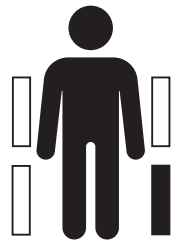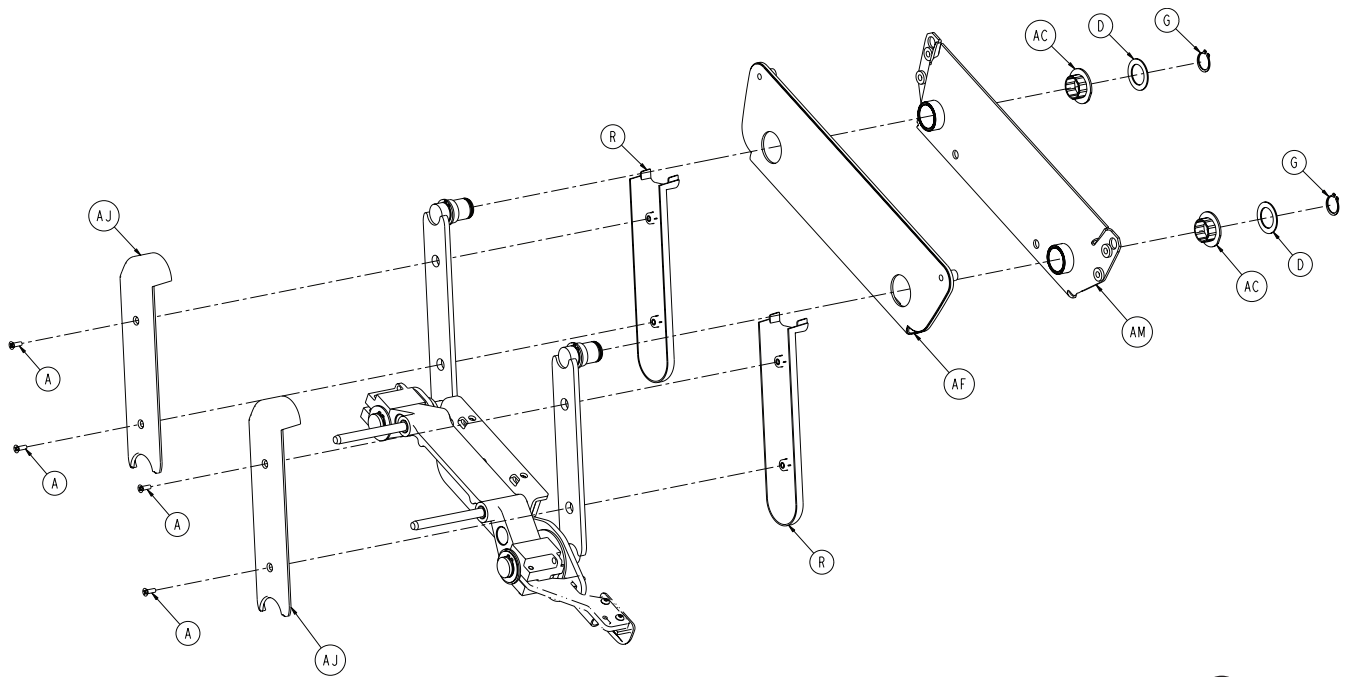

2031-400-305 / 2031-400-405


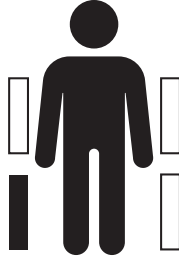


2031-400-305

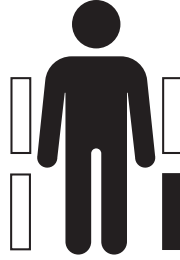


2031-400-405


2031-400-305			2031-400-405		
A	0001-072-000	4	A	0001-072-000	4
B	0003-344-000	2	B	0003-344-000	2
C	0011-185-000	1	C	0011-185-000	1
D	0011-343-000	3	D	0011-343-000	3
E	0014-093-000	2	E	0014-093-000	2
F	0023-090-000	4	F	0023-090-000	4
G	0028-128-000	6	G	0028-128-000	6
H	3000-200-334	1	H	3000-200-334	1
J	3000-400-513	2	J	3000-400-513	2
K	3000-400-523	2	K	3000-400-523	2
L	3000-400-557	4	L	3000-400-557	4
M	3001-400-527	1	M	3001-400-527	1
N	3001-400-555	1	N	3001-400-555	1
P	3001-400-564	2	P	3001-400-564	2
R	3001-400-619	2	R	3001-400-619	2
V	3002-400-501	1	V	3002-400-501	1
Y	3002-400-509	1	Y	3002-400-509	1
AA	3002-400-511	1	AA	3002-400-511	1
AB	3002-400-512	2	AB	3002-400-512	2
AC	3002-400-513	6	AC	3002-400-513	6
AD	3002-400-519	1	AD	3002-400-519	1
AE	3003-400-528	1	AE	3003-400-528	1
AF	3003-400-526	1	AF	3003-400-526	1
AG	2032-401-030	1	AG	2032-401-025	1
AH	3003-400-520	1	AH	3003-400-520	1
AJ	5000-020-005	2	AJ	5000-020-005	2
AK	0003-226-000	4	AK	0003-226-000	4
AL	3001-400-558	4	AL	3001-400-558	4
AM	3003-400-580	1	AM	3003-400-580	1

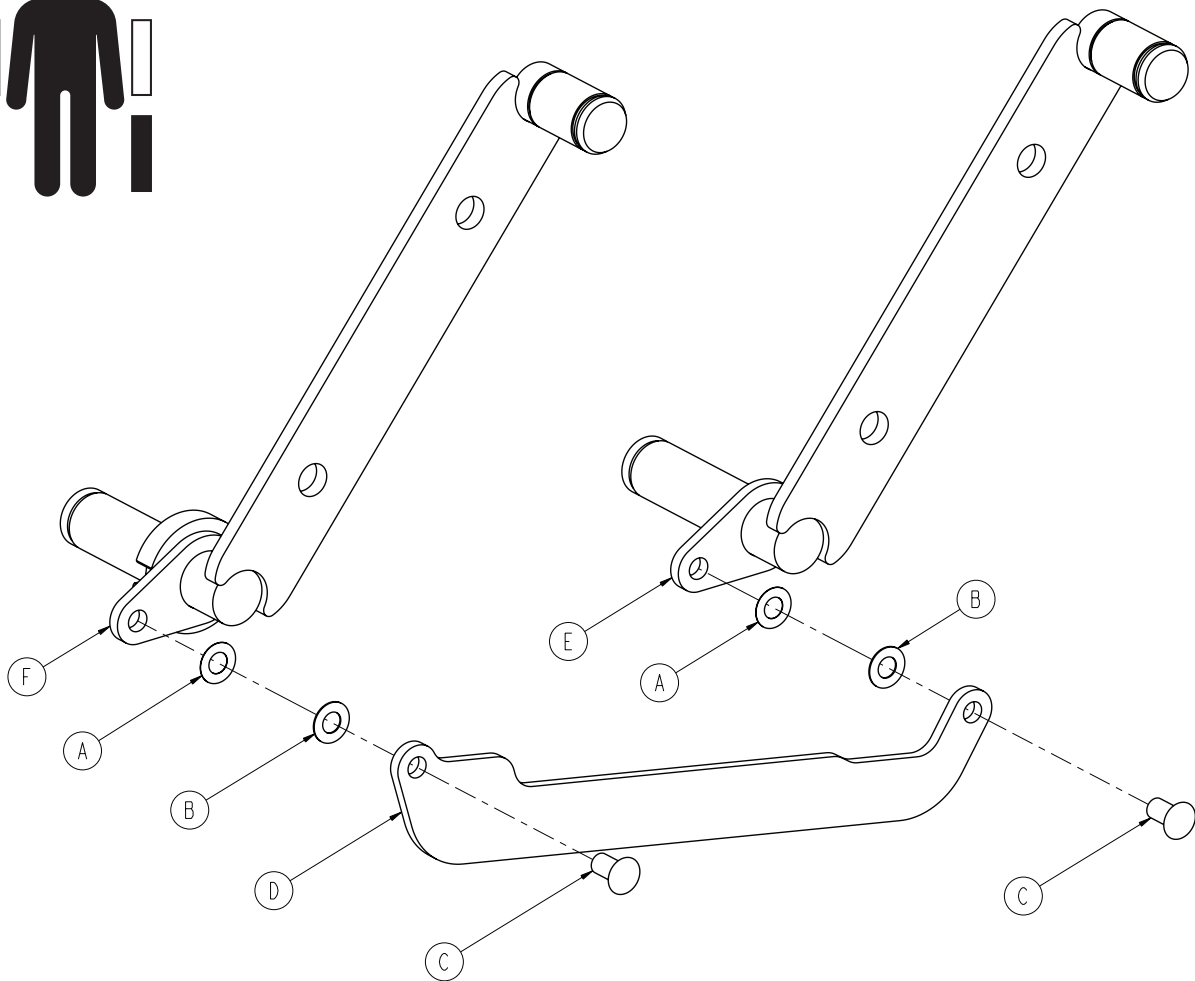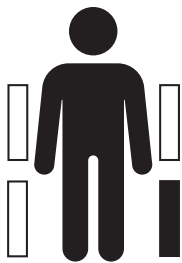


A	0011-377-000	2
B	0011-491-000	2
C	3001-400-501	2
D	3003-400-011	1
E	3003-401-327	1
F	2032-401-328	1

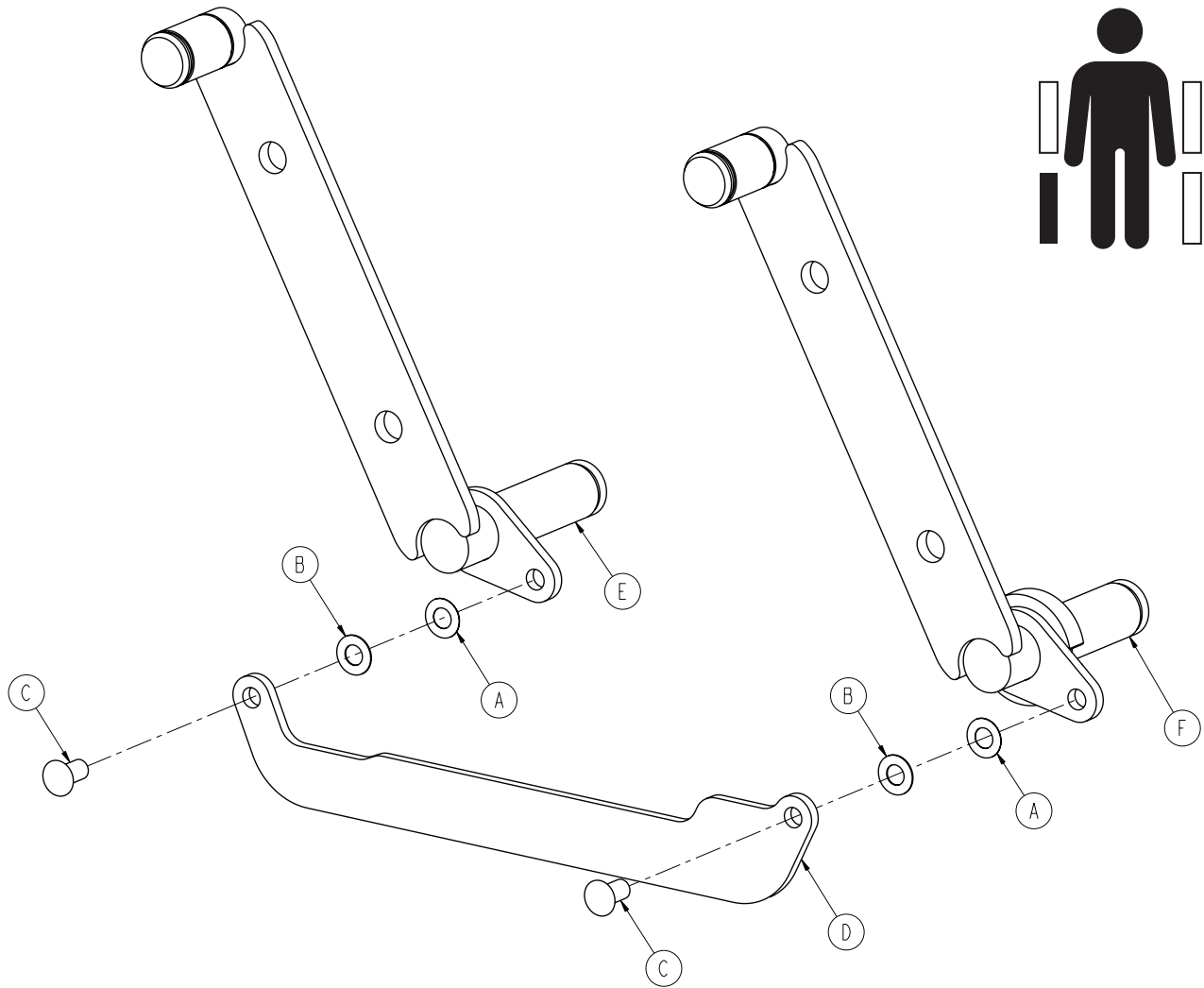


A	0011-377-000	2
B	0011-491-000	2
C	3001-400-501	2
D	3003-400-011	1
E	3003-401-427	1
F	2032-401-428	1

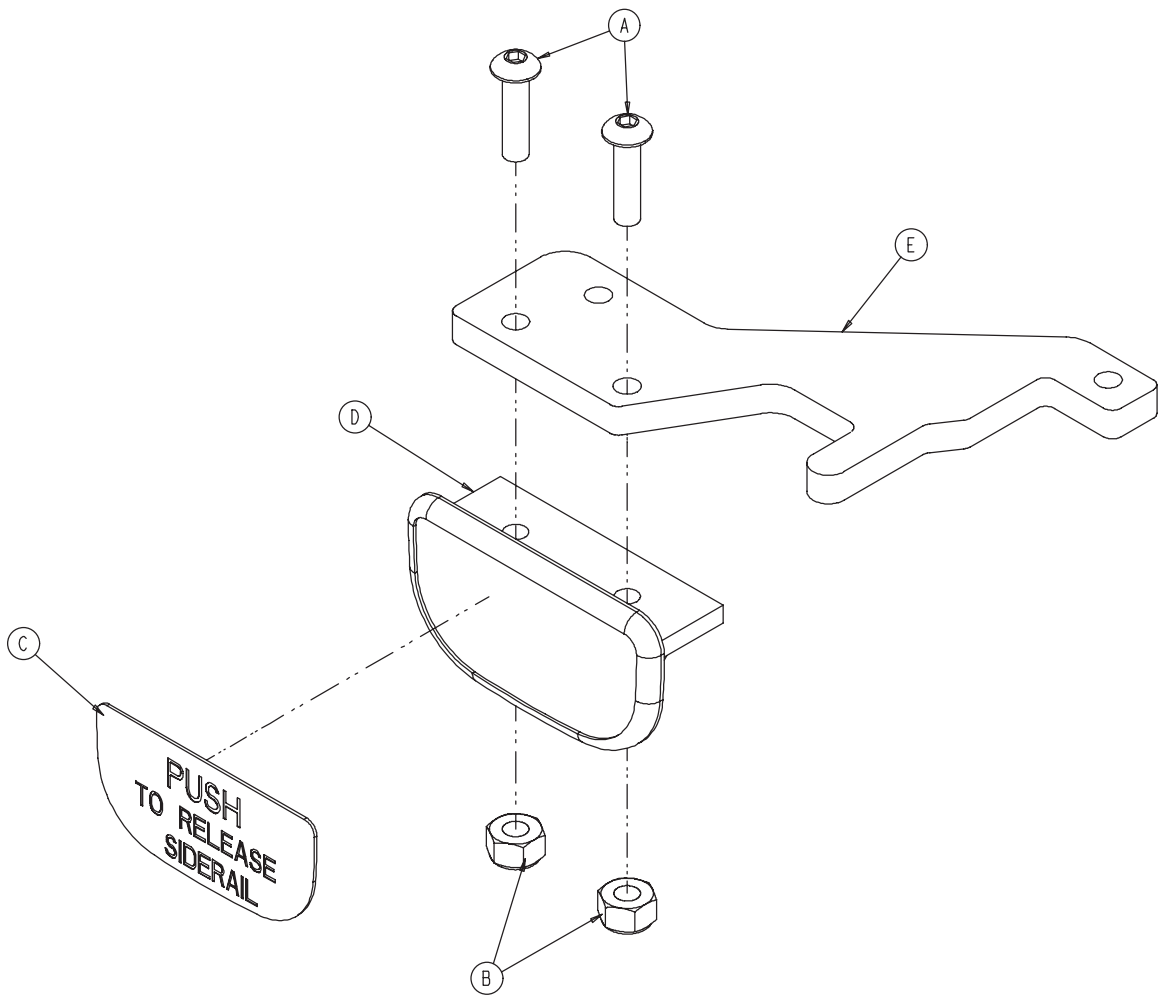


A	0004-278-000	2
B	0016-002-000	2
C	3003-503-901	1
	2031-006-016 (Polish)	
	2031-008-016 (Portuguese)	
D	3001-400-514	1
E	3002-400-510	1

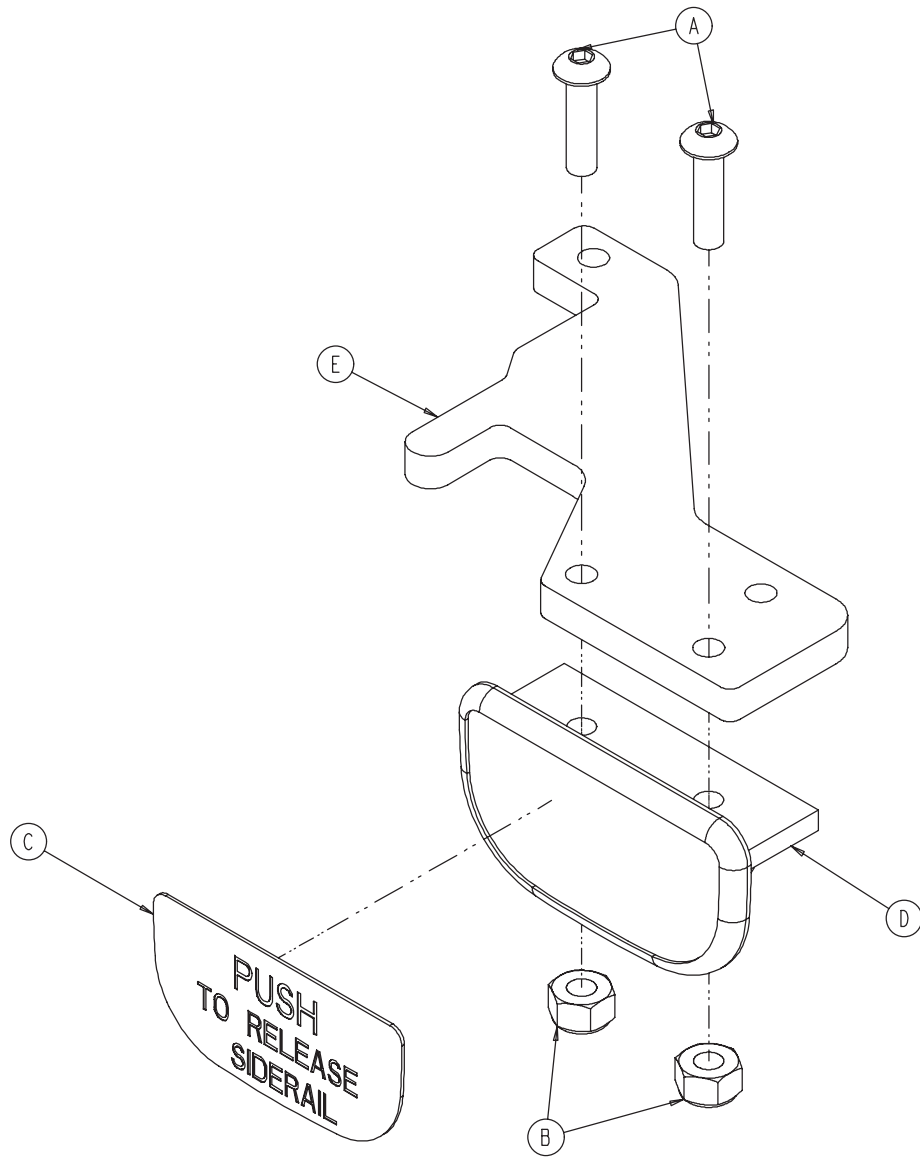


A	0004-278-000	2
B	0016-002-000	2
C	3003-503-901	1
	2031-006-016 (Polish)	
	2031-008-016 (Portuguese)	
D	3001-400-514	1
E	3002-400-510	1

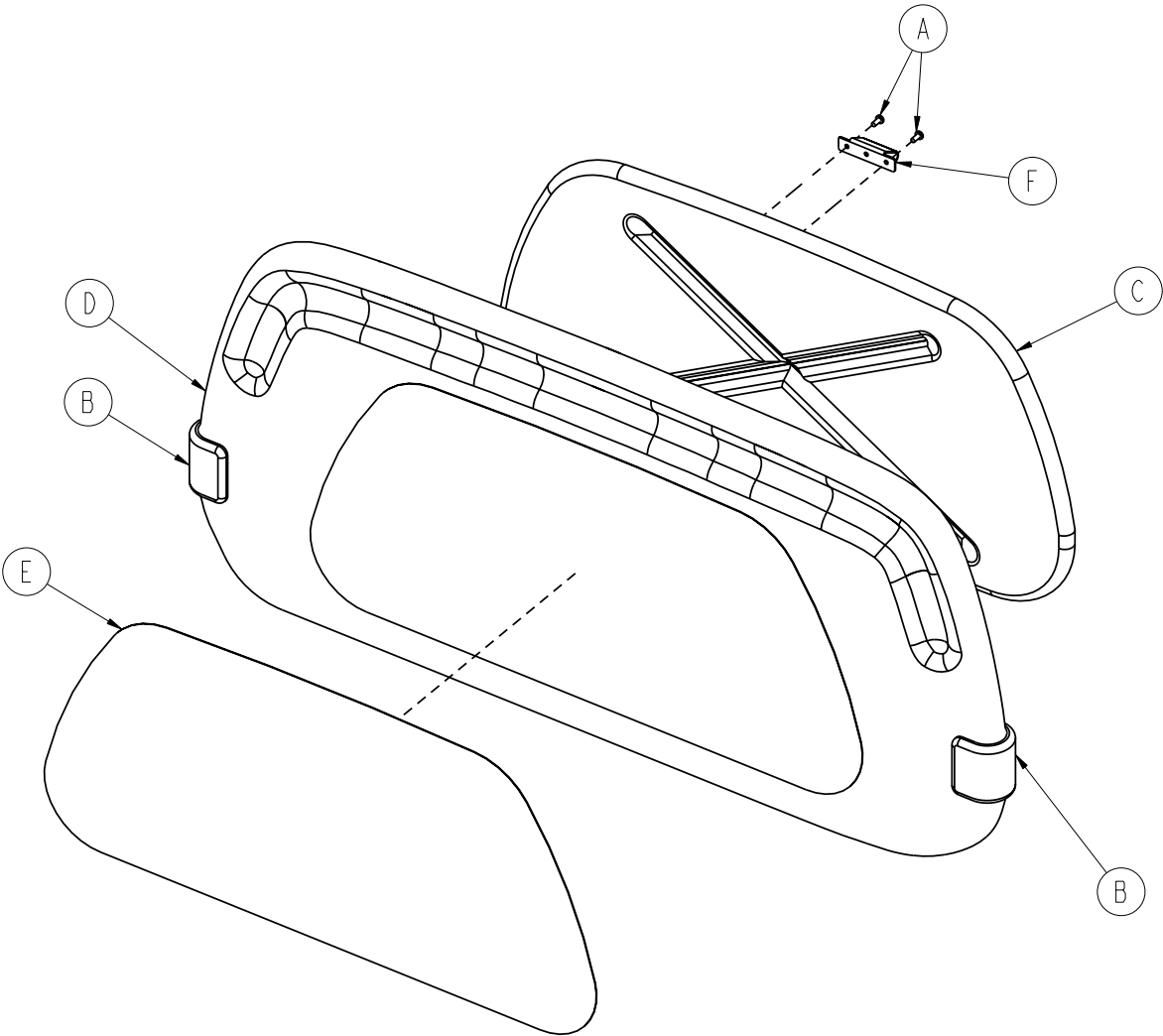


A	0023-088-000	2
B	2035-500-007	2
C	3000-526-001	1
D	3000-600-010	1
E	3000-600-056	1
F	3006-526-002	1

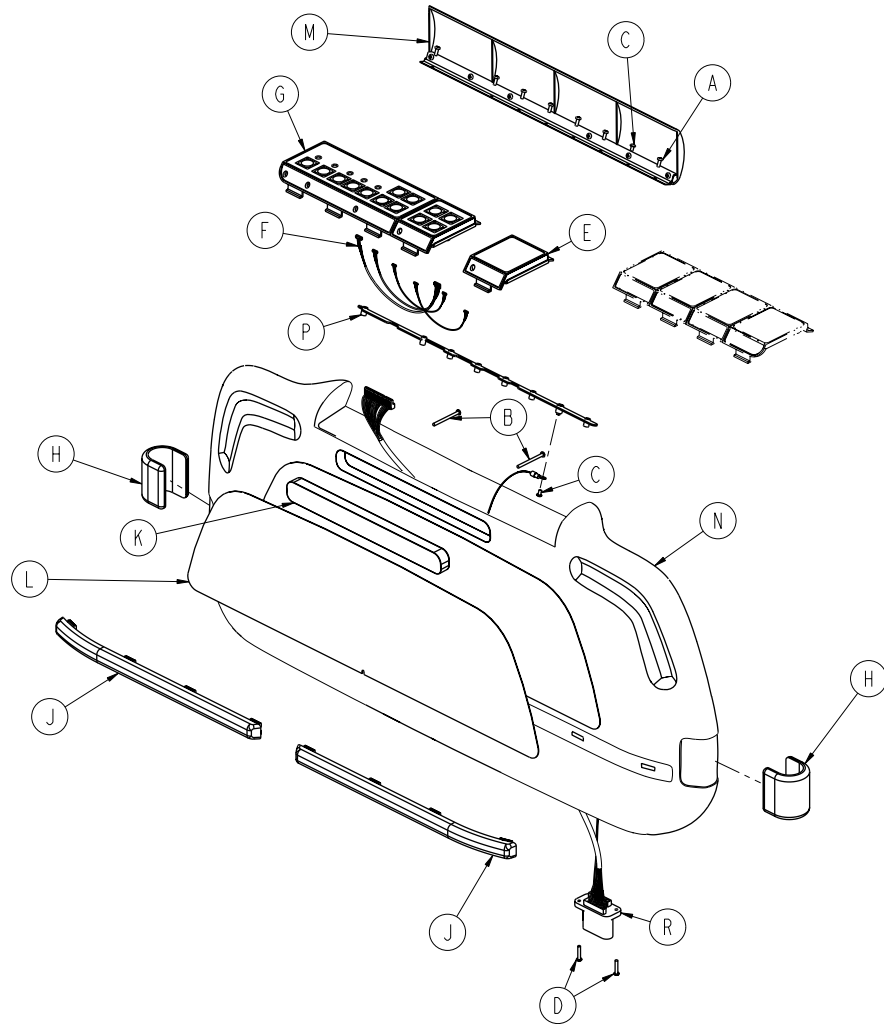


A	0023-103-000	7
B	0023-118-000	2
C	0050-038-000	2
D	0050-039-000	2
E	2025-136-021	1
F	2025-136-801	1
G	2032-235-020	1
H	2035-500-007	2
J	2035-500-008	2
K	3000-500-008	1
L	3000-500-056	1
M	3001-500-001	1
N	3001-500-010	1
P	3001-500-064	1
R	3001-500-801	1

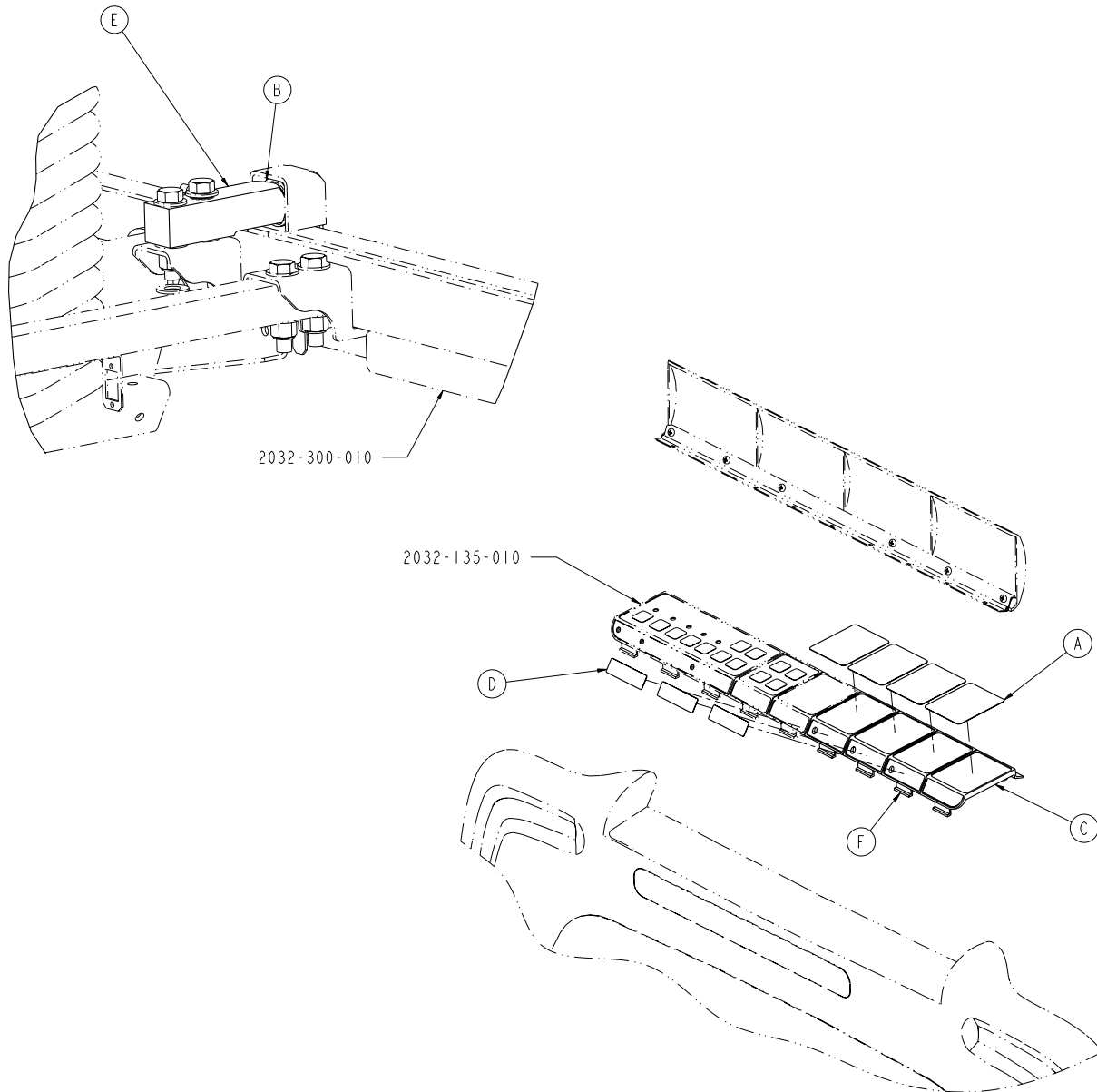


A	2035-500-101	4
B	3000-300-353	4
C	3000-500-004	1
D	3000-500-026	3
E	3001-300-511	4
F	3001-500-003	3

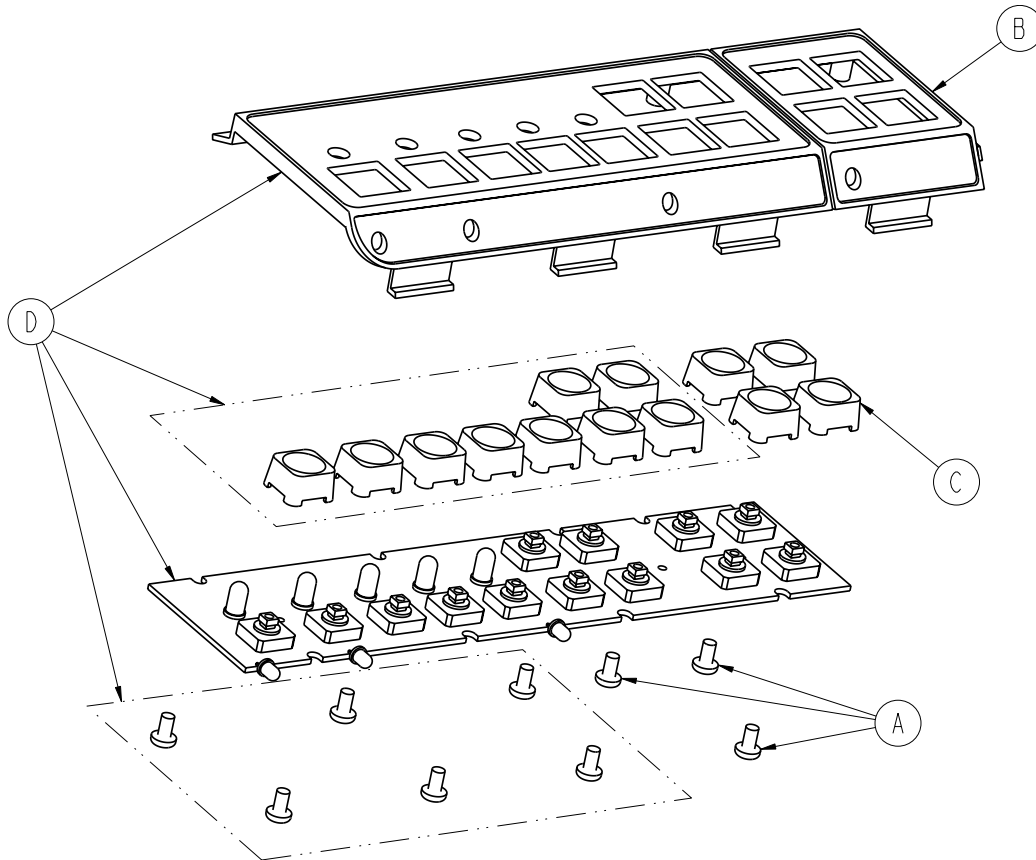


A	0023-087-000	3
B	3000-501-001	1
C	3001-400-953	4
D	3001-500-028	1

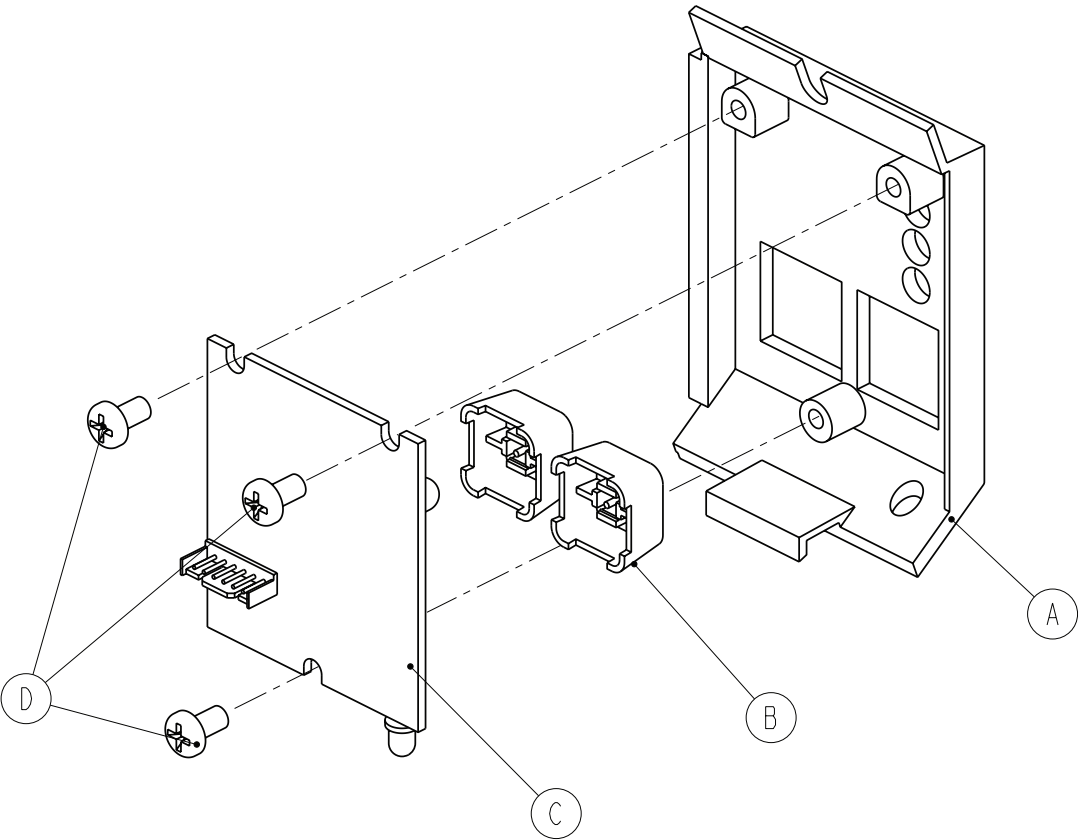


A	3000-508-001	1
B	3001-400-953	2
C	2025-136-900	1
D	0023-087-000	3

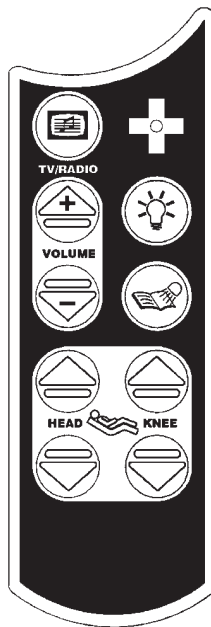


3001-315-012

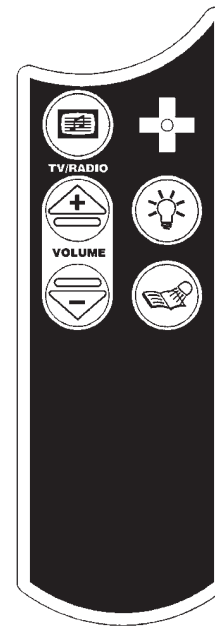


3001-315-016

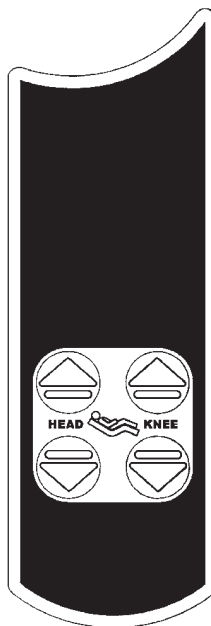



3001-315-014

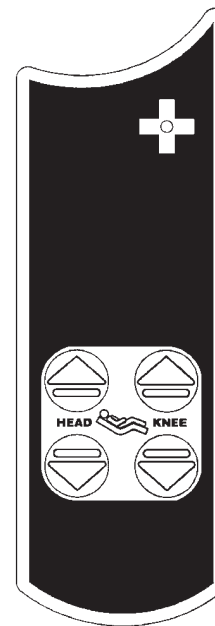

3001-315-018



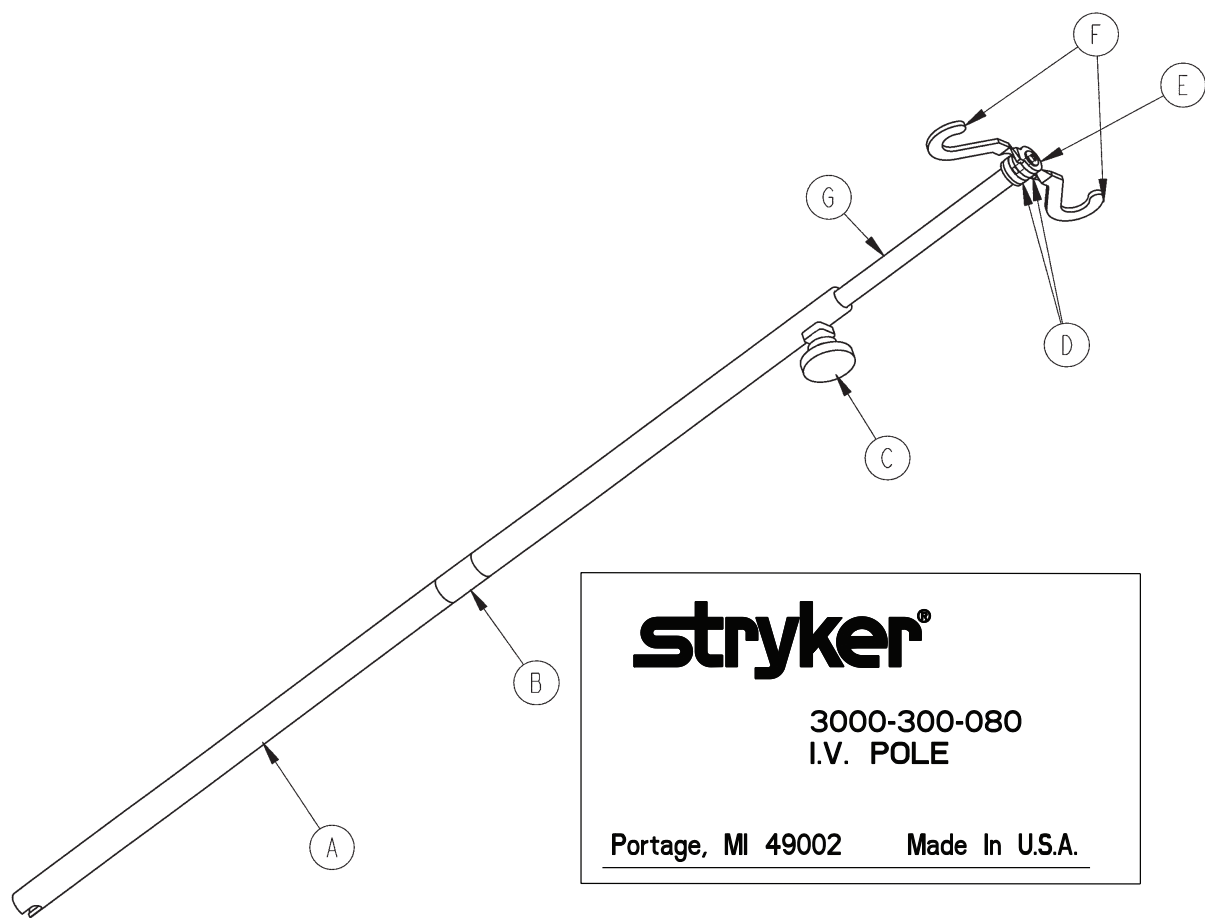


A	3000-300-081	1
B	3000-300-089	1
	2031-006-031 (Polish)	
	2031-008-031 (Portuguese)	
C	0024-050-000	1
D	0052-017-000	2
E	0007-040-000	1
F	1010-059-016	2
G	3000-300-085	1

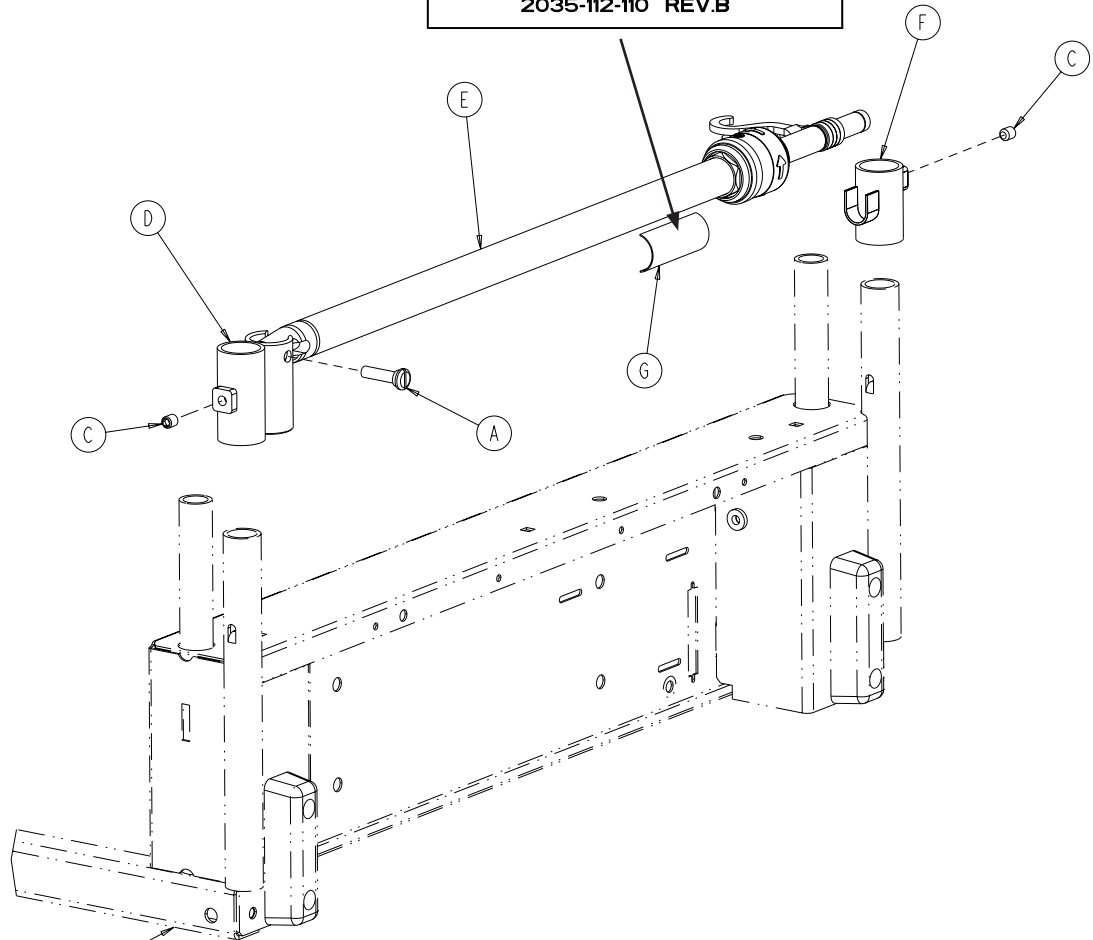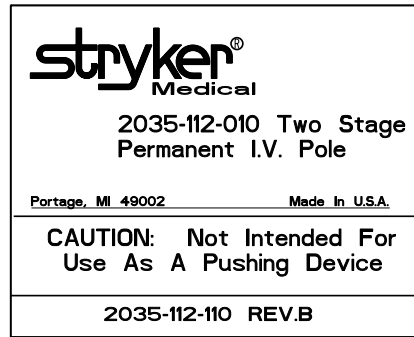


2035-031-054

A	0785-048-025	1
C	0021-193-000	2
D	2035-112-011	1
E	2035-112-010	1
F	3000-311-036	1
G	2035-112-110	1
	2031-006-029 (Polish)	
	2031-008-029 (Portuguese)	
H	0025-079-000	1

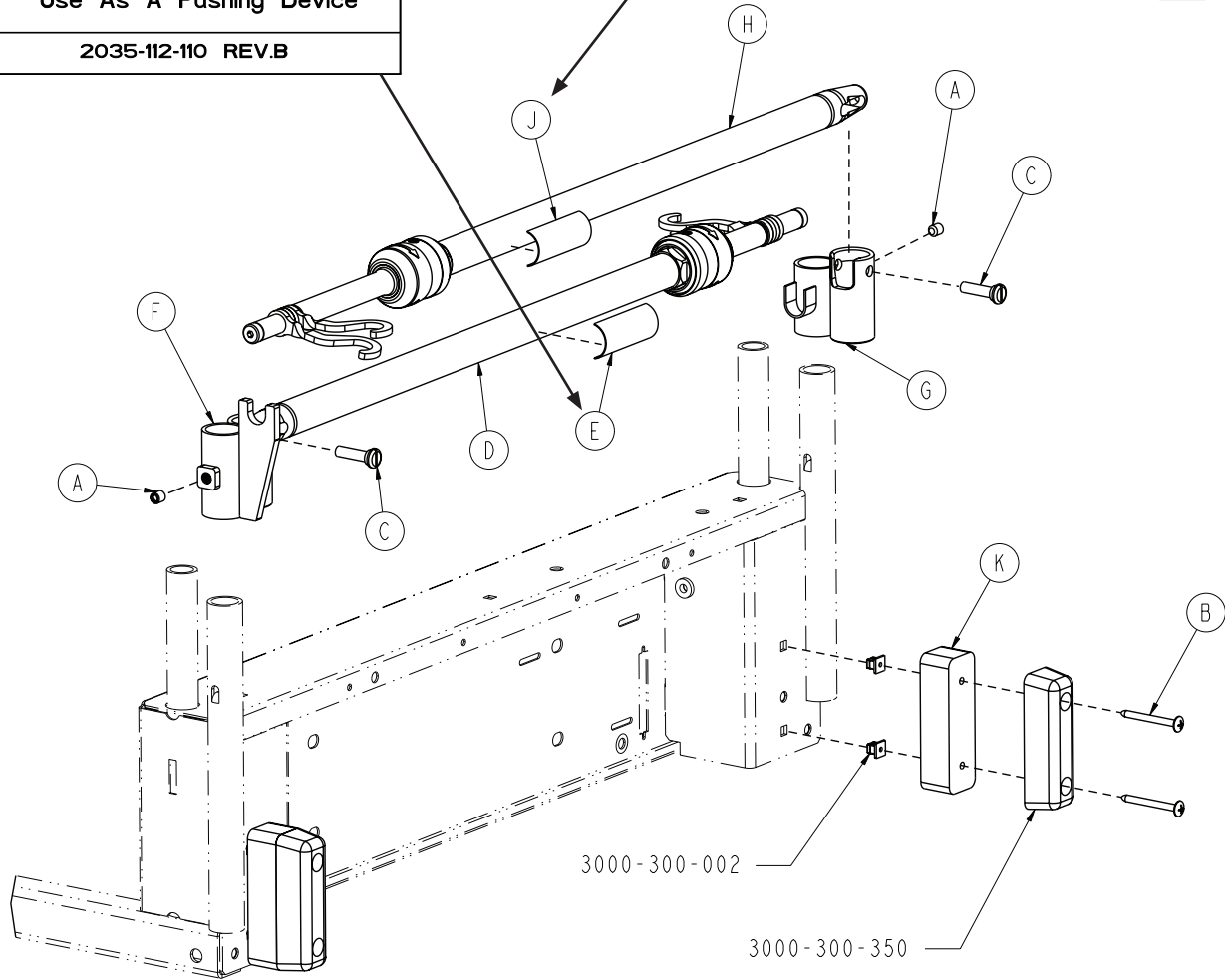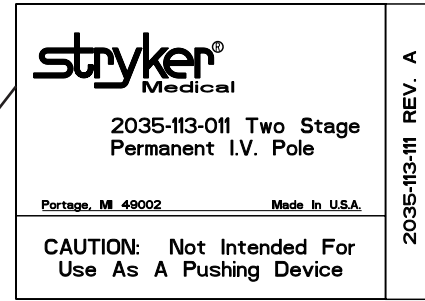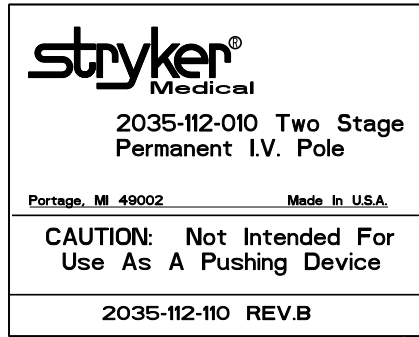


A	0021-193-000	2
B	0023-277-000	4
C	0785-048-025	2
D	2035-112-010	1
E	2035-112-110	
	2031-006-029 (Polish)	
	2031-008-029 (Portuguese)	1
F	2035-113-021	1
G	2035-113-012	1
H	2035-113-011	1
J	2035-113-111	
	2031-006-030 (Polish)	
	2031-008-030 (Portuguese)	1
K	2035-113-006	2

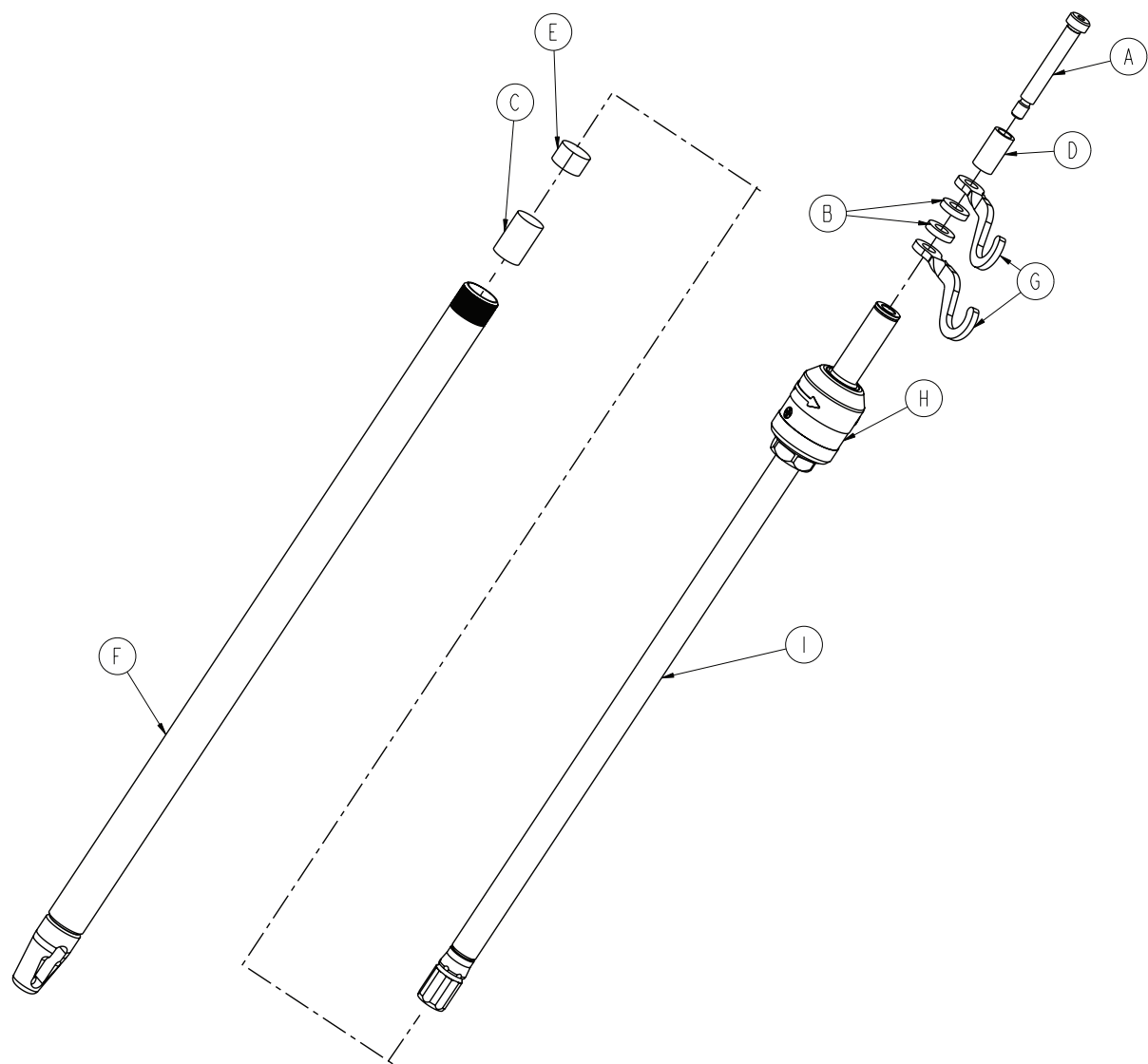


2035-112-010		
A	0008-031-000	1
B	0052-017-000	2
C	0052-310-000	1
D	0926-400-062	1
E	1001-259-013	1
F	1001-259-032	1
G	1010-059-016	2
H	0785-035-103	1
I	1211-110-029	1


2035-113-011		
A	0008-031-000	1
B	0052-017-000	2
C	0052-311-000	1
D	0926-400-062	1
E	1001-259-013	1
F	1001-259-032	1
G	1010-059-016	2
H	0785-035-103	1
I	1211-110-029	1

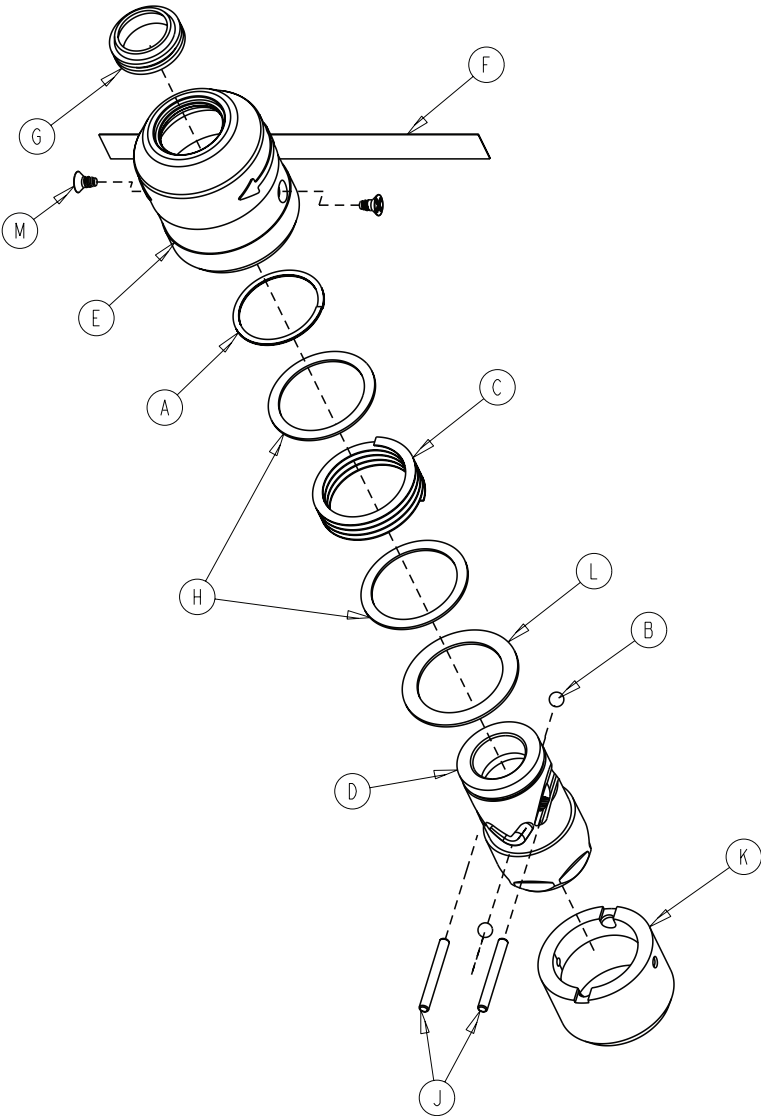


A	0028-167-000	1
B	0031-004-000	2
C	0038-392-000	1
D	0785-035-023	1
E	0785-035-024	1
F	0785-035-029	1
G	1211-110-018	1
H	1211-110-020	2
J	1211-110-021	2
K	1211-110-022	1
L	1211-110-035	1
M	1211-110-036	2

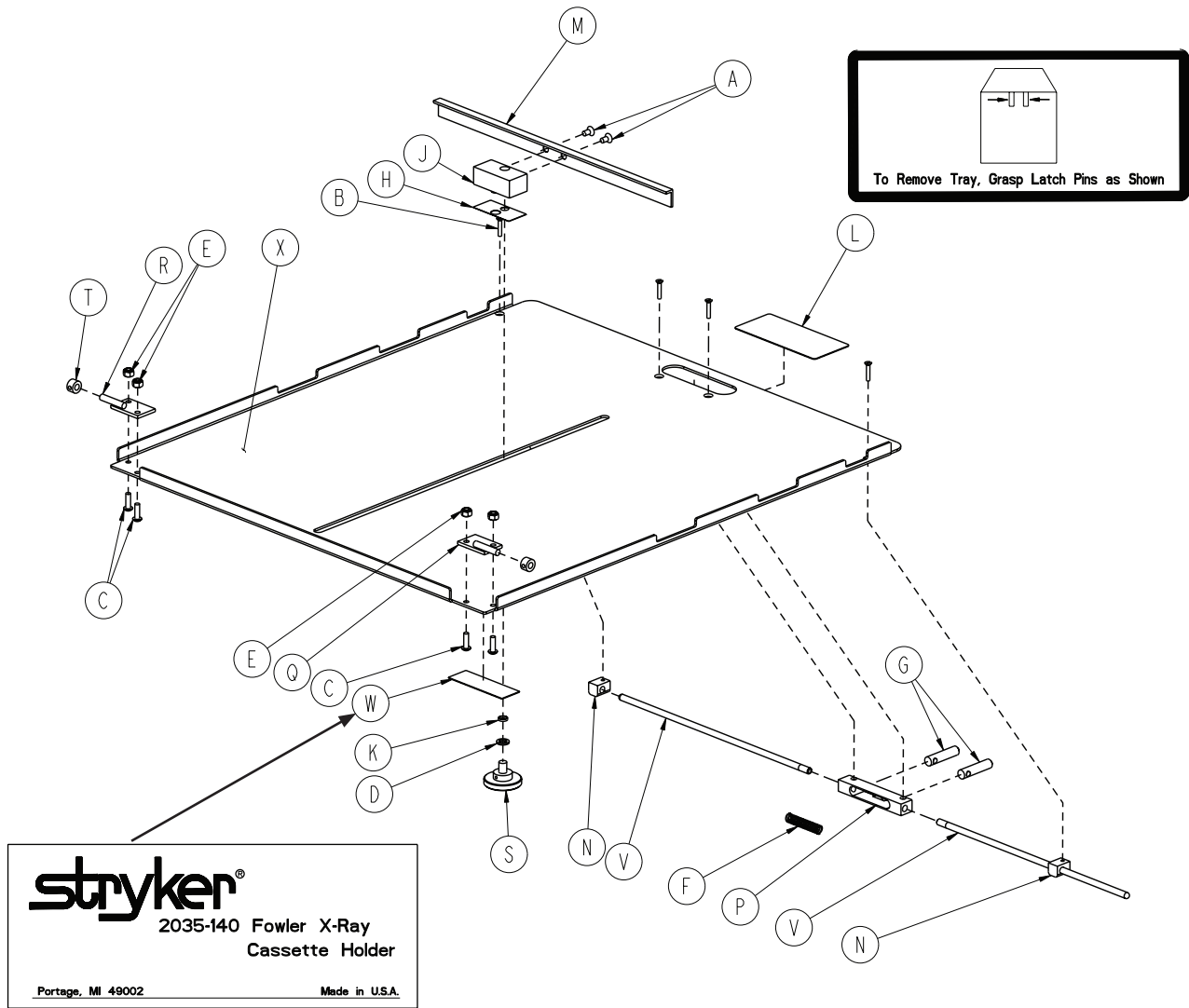


A	0001-020-000	2	M	2025-023-028	1
B	0004-049-000	4	N	1010-023-037	2
C	0004-149-000	4	P	1020-023-016	1
D	0014-003-000	1	Q	1020-023-019	1
E	0016-003-000	4	R	1020-023-020	1
F	0038-122-000	1	S	1020-023-021	1
G	0926-023-064	2	T	0042-013-000	2
H	0926-023-069	1	V	2025-140-002	2
J	0926-023-070	1	W	2035-140-025	1
K	0926-023-071	1		2031-006-033 (Polish)	
L	1010-023-019	1		2031-008-033 (Portuguese)	
	2031-006-032 (Polish)		X	2032-140-099	1
	2031-008-032 (Portuguese)				

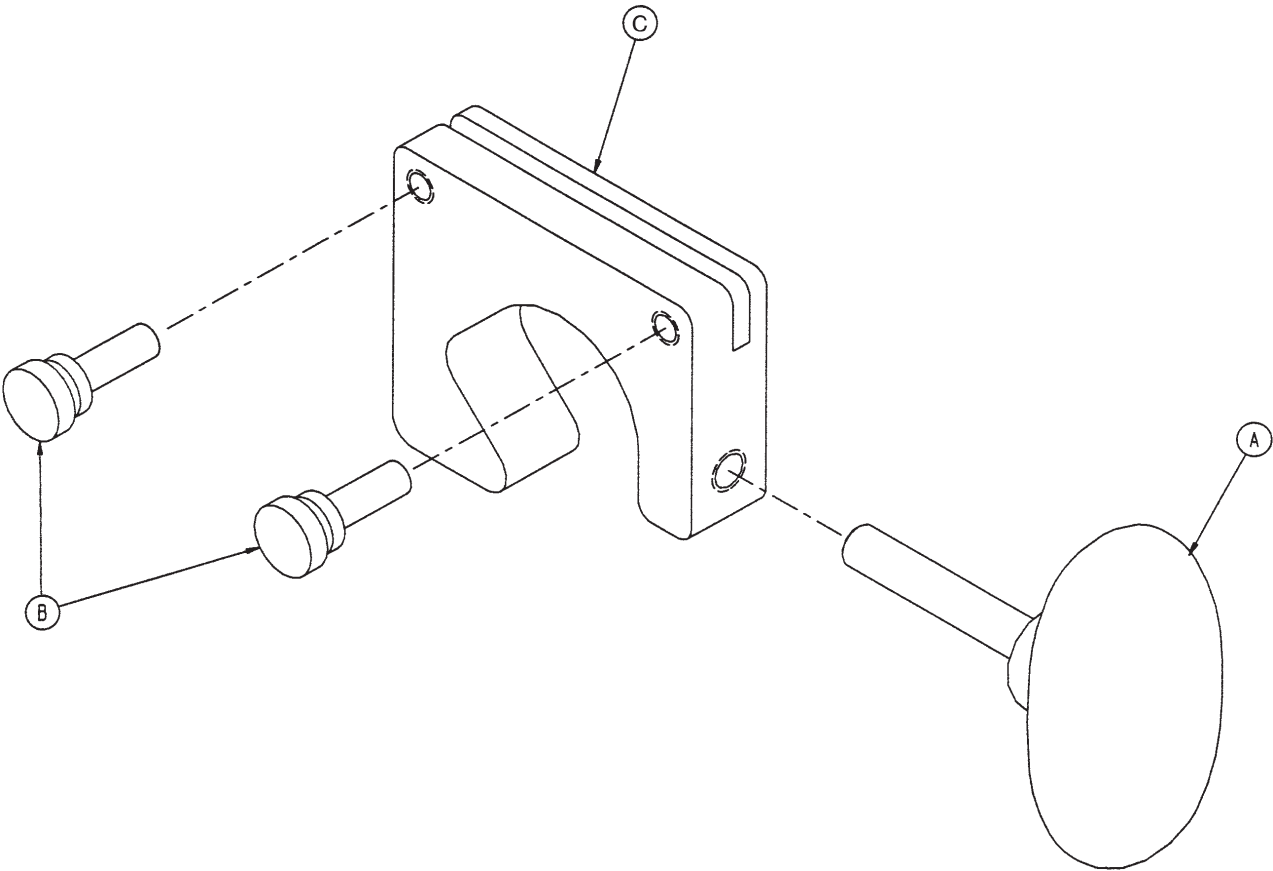


A	0024-063-000	1
B	0024-064-000	2
C	2035-018-011	1

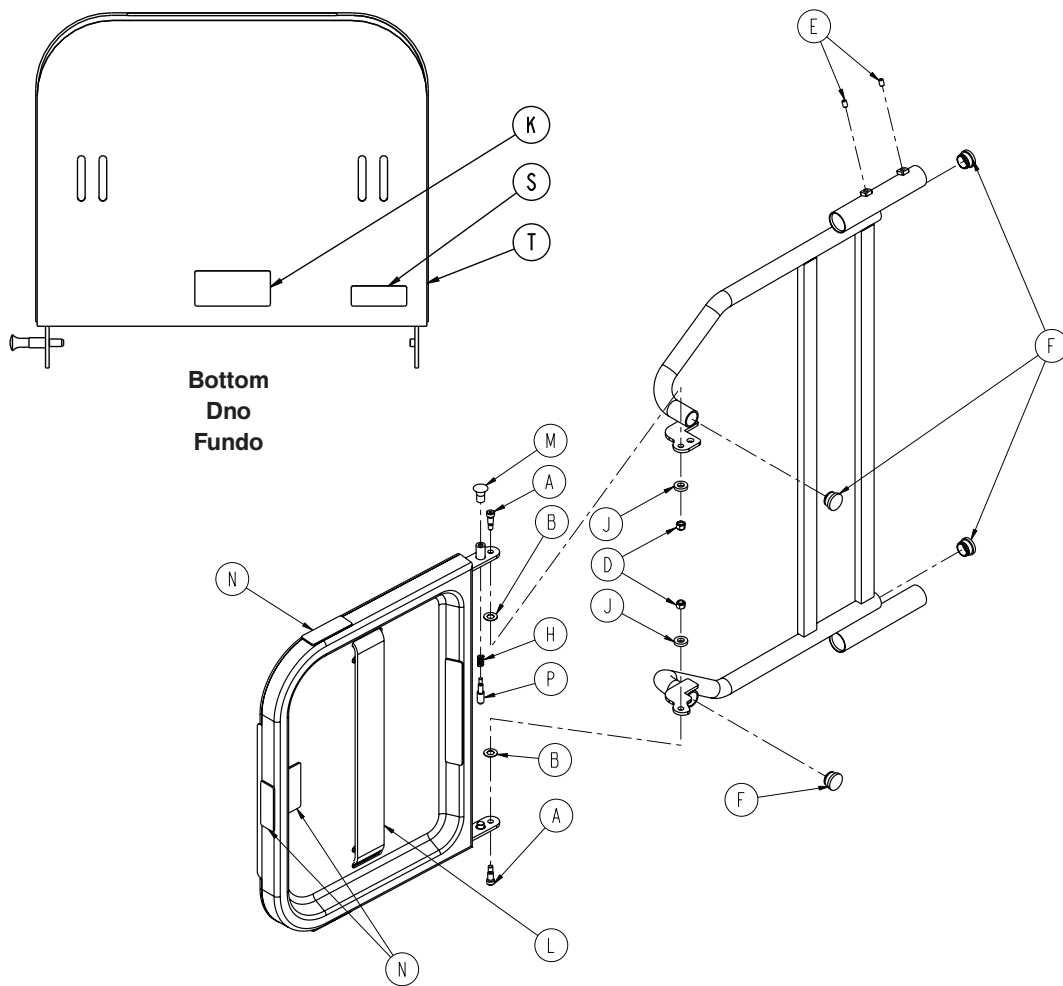


A	0008-049-000	2
B	0014-020-000	2
D	0016-028-000	2
E	0021-017-000	4
F	0037-214-000	4
H	0038-133-000	1
J	0052-017-000	2
K	1010-050-019	1
	2031-006-035 (Polish)	
	2031-008-035 (Portuguese)	
L	1010-050-021	1
M	1010-050-050	1
N	1010-050-057	4
	2031-006-036 (Polish)	
	2031-008-036 (Portuguese)	
P	1010-050-242	1
R	2025-120-005	1
	2031-006-037 (Polish)	
	2031-008-037 (Portuguese)	
S	2025-120-006	1
	2031-006-038 (Polish)	
	2031-008-038 (Portuguese)	
T	2025-120-018	1
W	2025-120-025	1

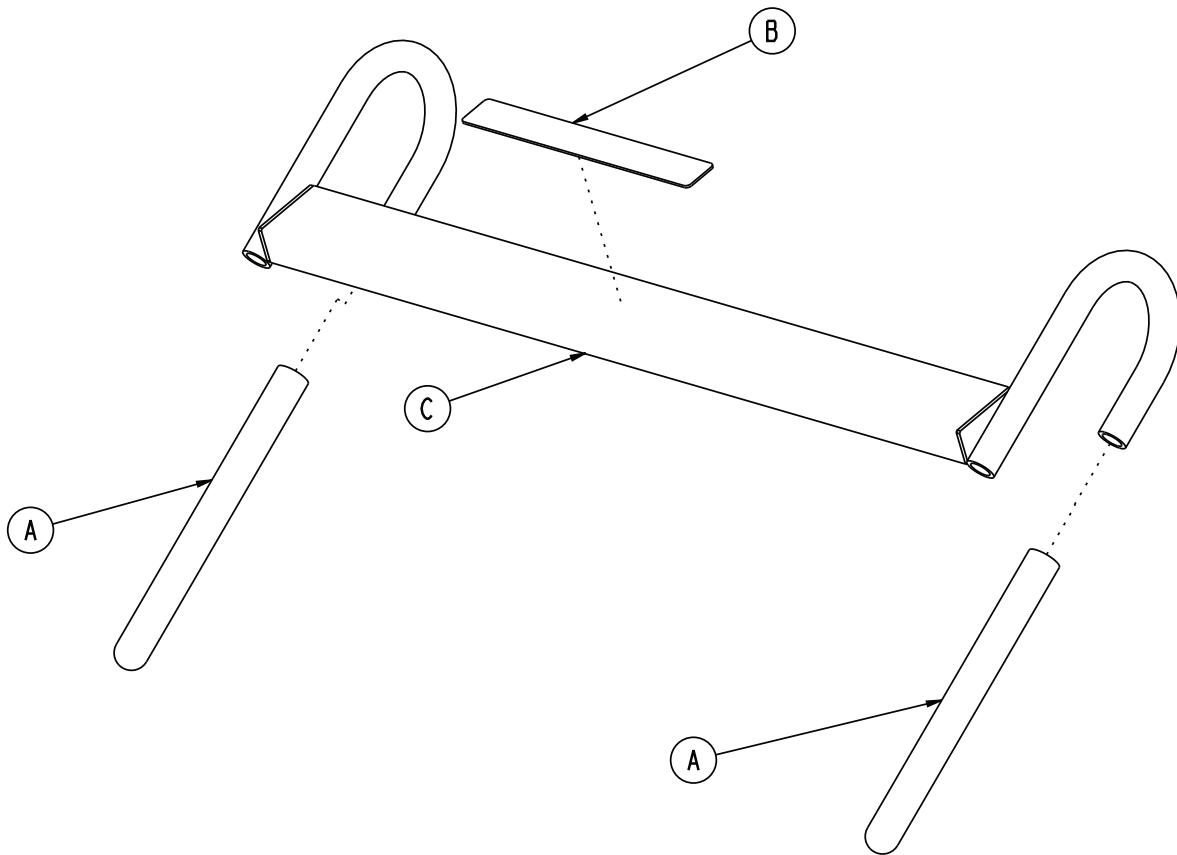


A	0058-087-000	2
B	2030-140-002	1
	2031-006-039 (Polish)	
	2031-008-039 (Portuguese)	
C	2040-111-005	1

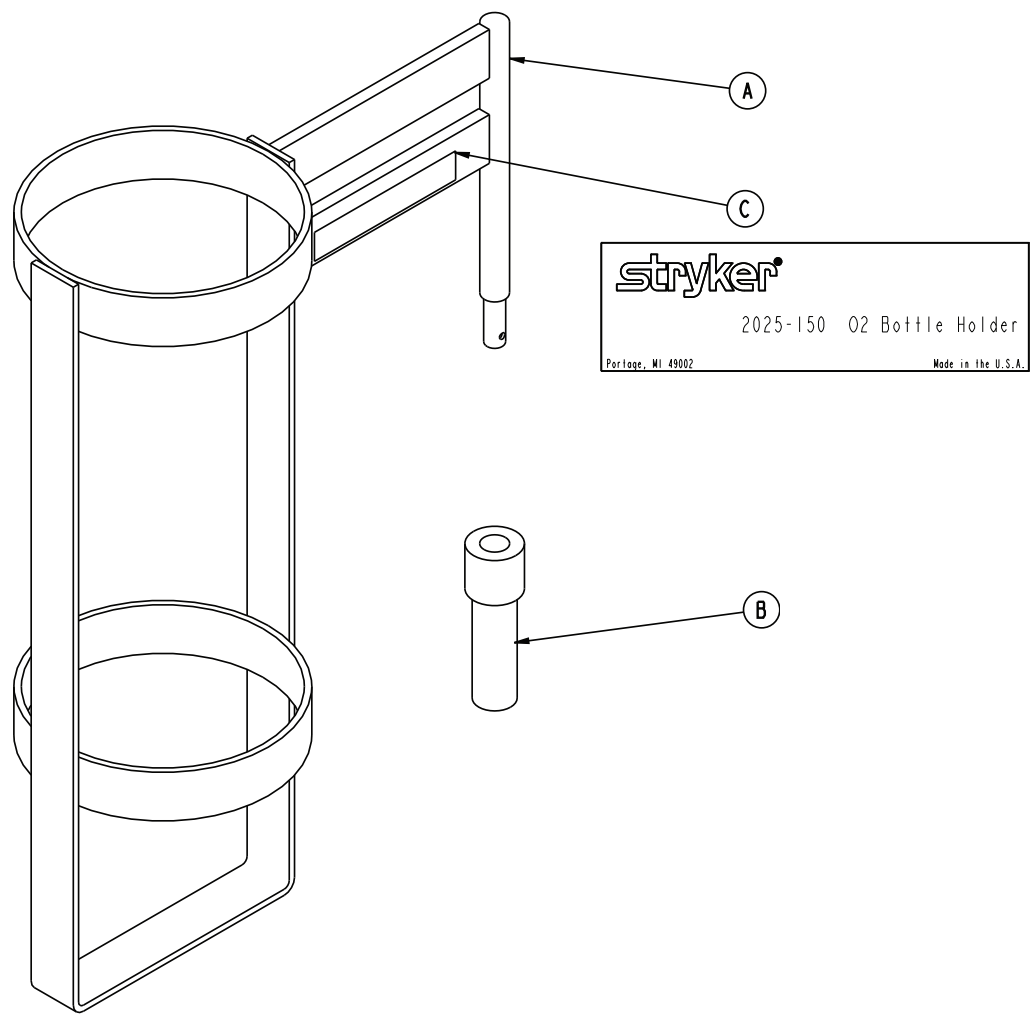

A	3002-045-001	1
B	2025-150-001	1
C	2025-150-002	1
	2031-006-034 (Polish)	
	2031-008-034 (Portuguese)	

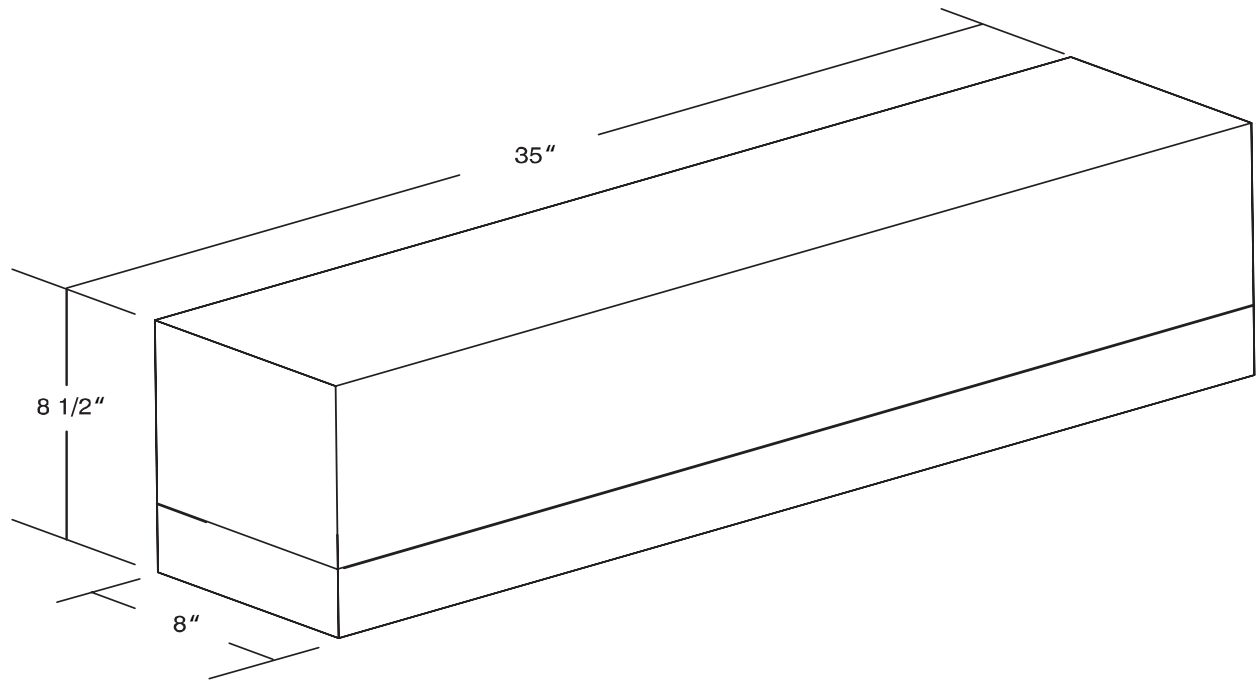


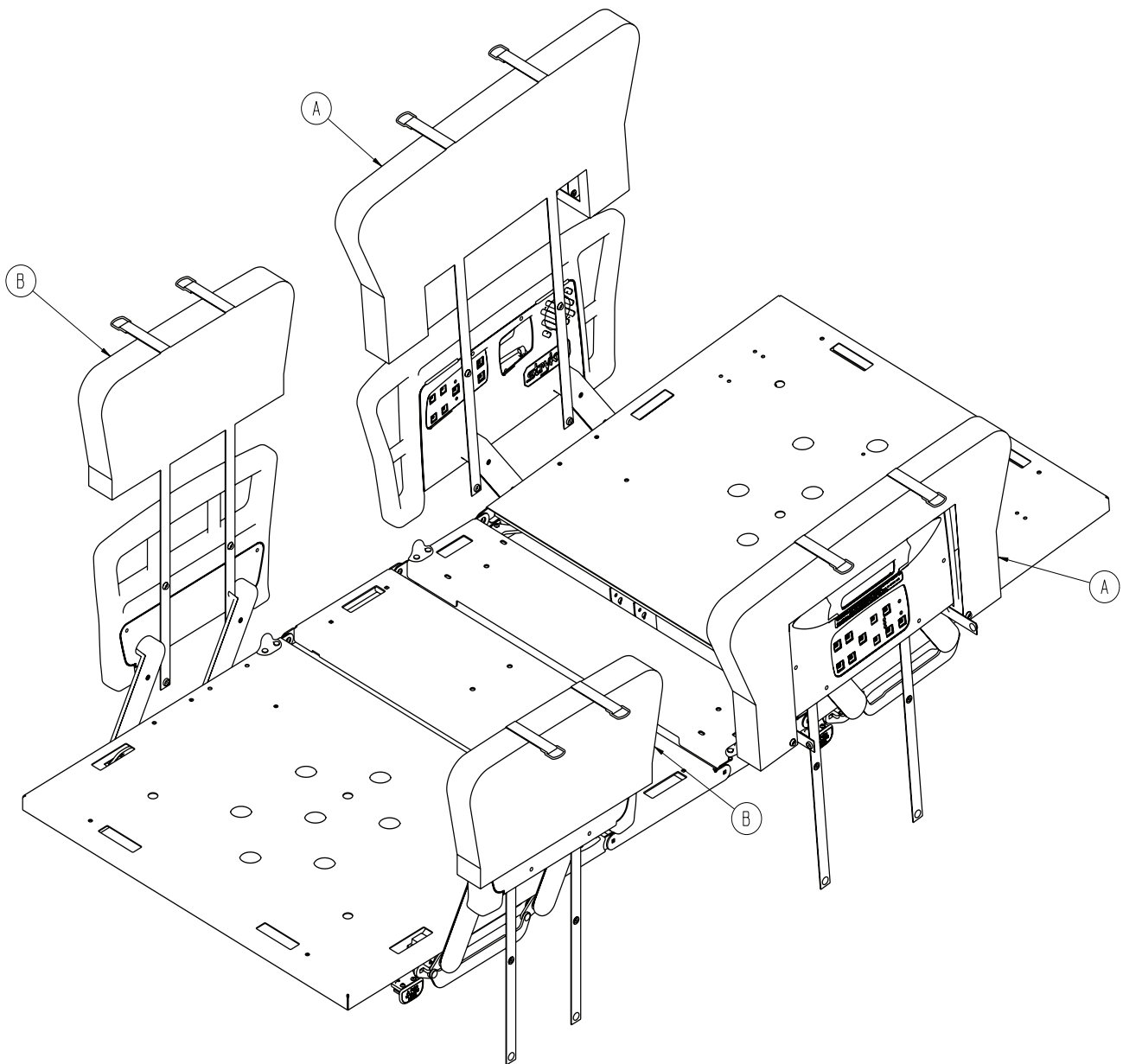

A	3003-336-011	2
B	3003-336-013	2

Recycling Passport


2031-400-105 / 2031-400-205

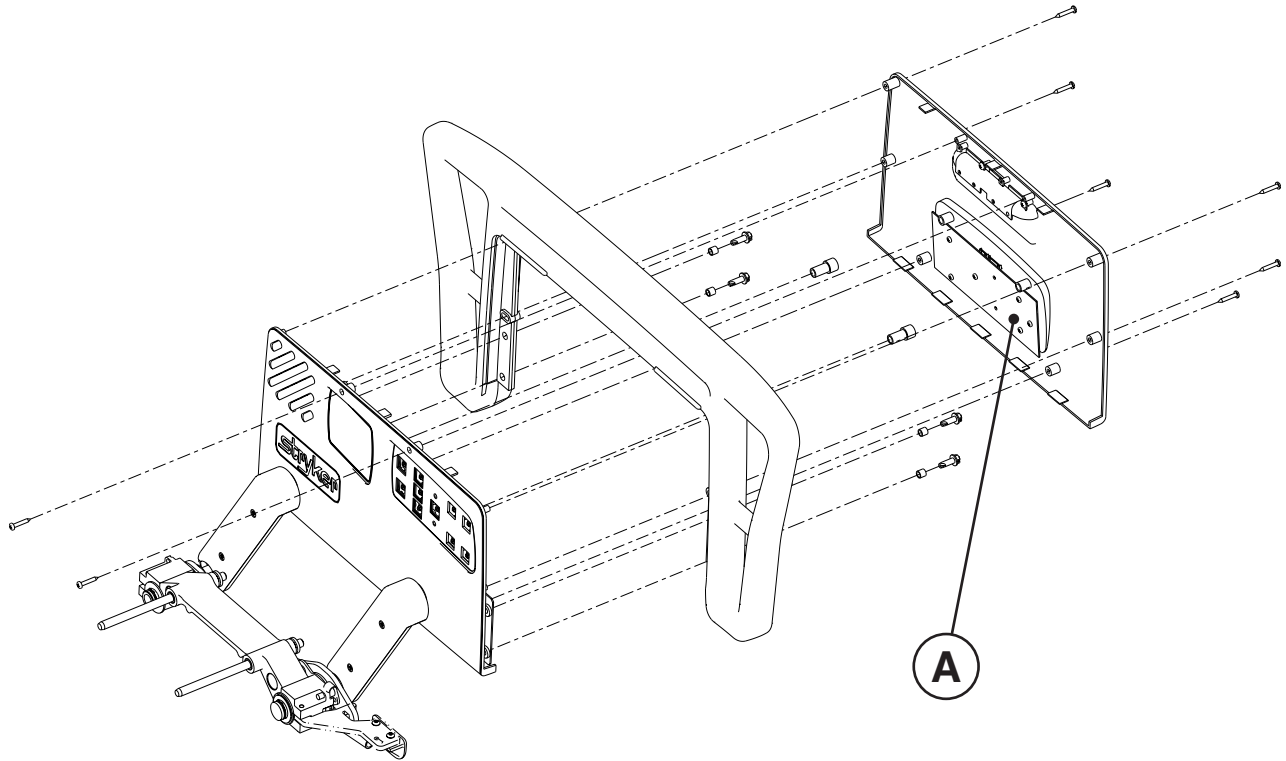

A	2035-400-900	2
---	--------------	---

Recycling Passport


2031-400-105 & 2031-400-205

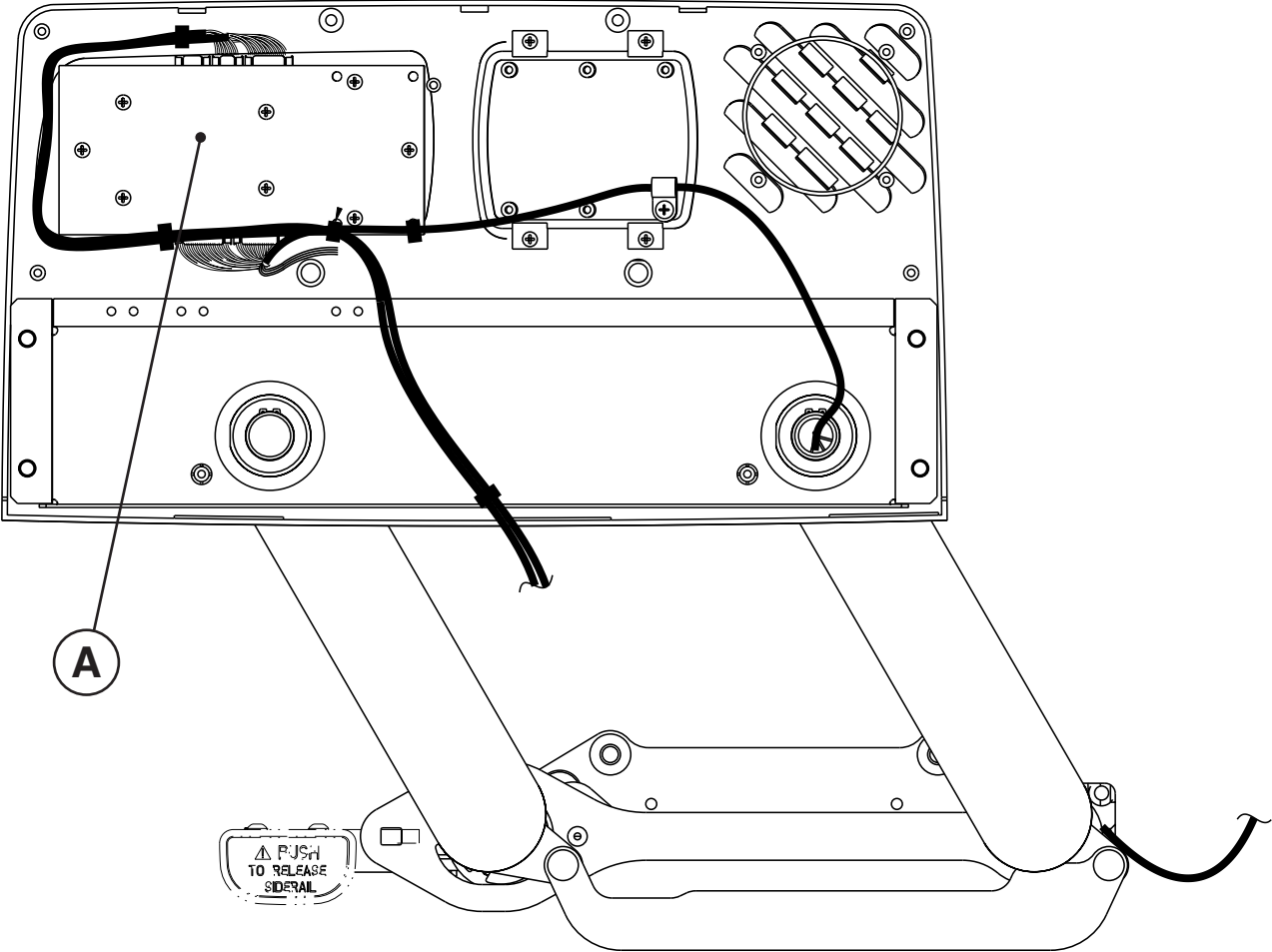

A	3001-400-930	2
---	--------------	---

Recycling Passport


2032-135-010

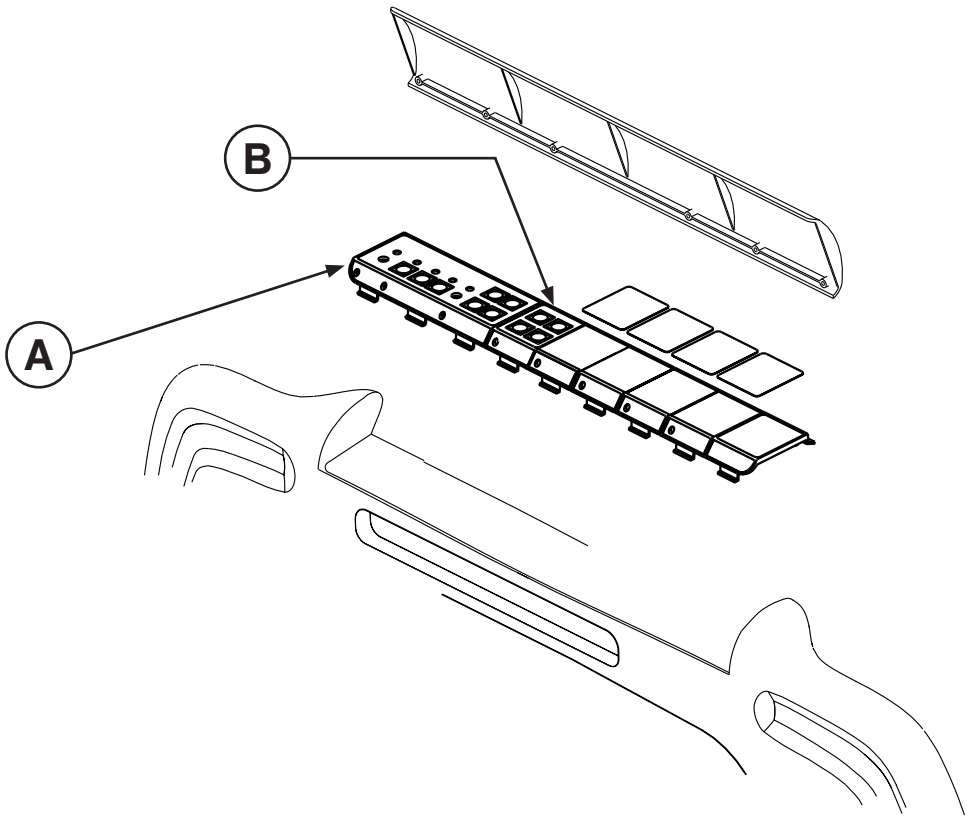

A	2032-235-010	1
B	2025-136-021	1

Recycling Passport


2031-030-201 / 2031-030-205

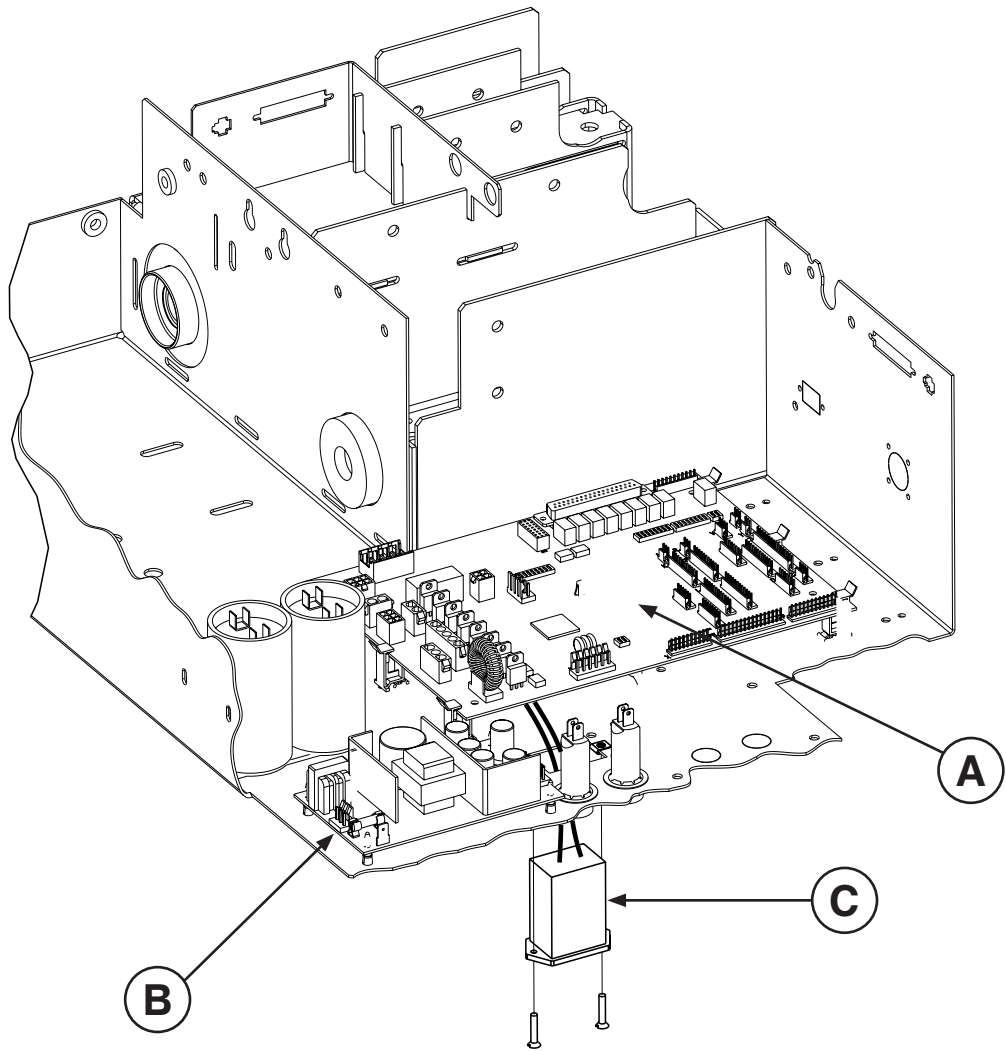

A	3002-407-950	1
B	0059-157-000	1
C	3000-303-871	1

Recycling Passport

2031-030-204

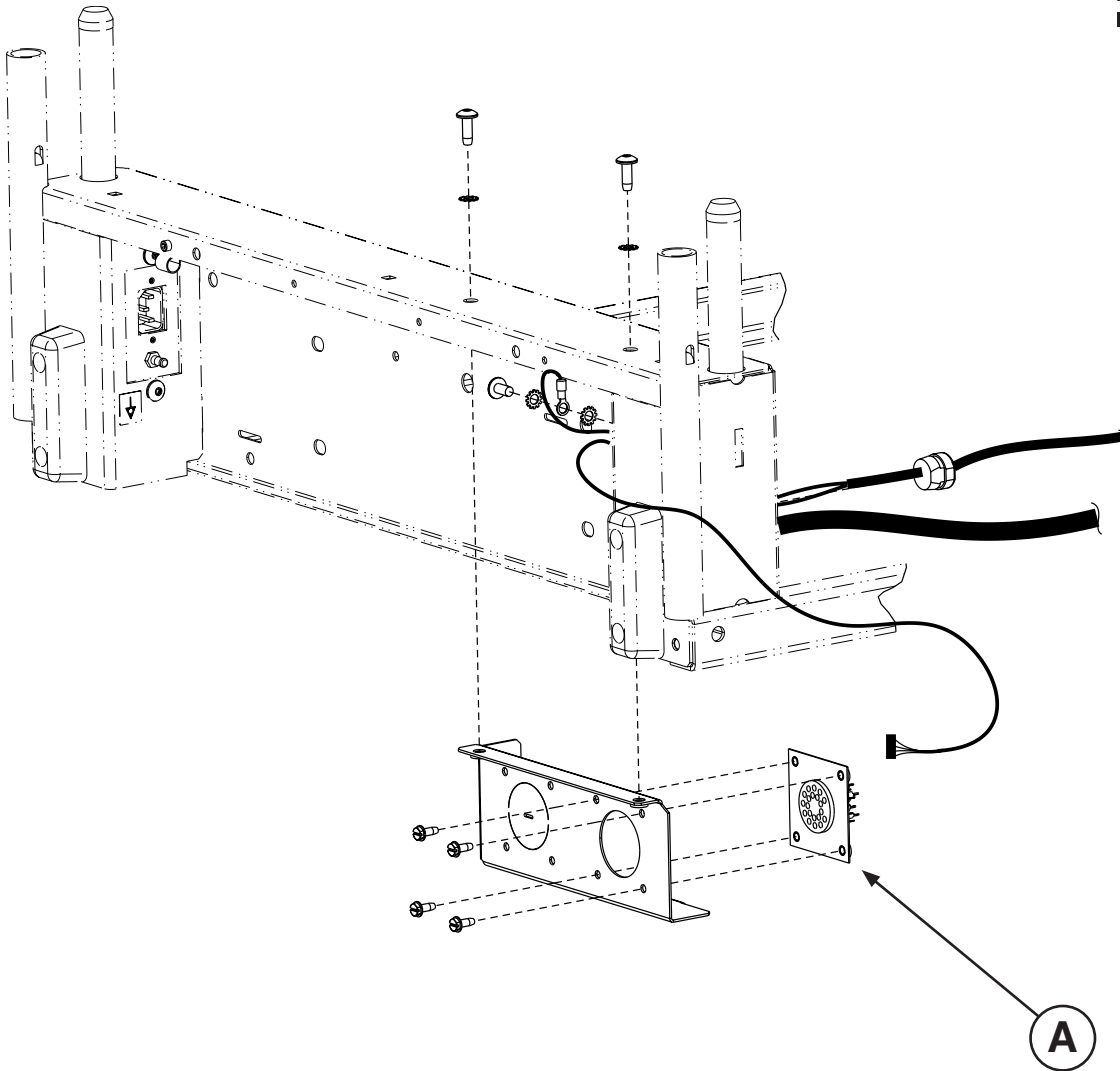

A	3001-314-920	1
---	--------------	---



Stryker Medical
3800 E. Centre Ave.
Portage, Michigan 49002
USA



Stryker EMEA RA/QA Director
ZAC - Avenue de Satolas Green
69881 MEYZIEU Cedex
France



stryker[®]