

QQ-30500
IVPOLE1 – IV POLE, 2 PRONG

Product: - CHAMPION RECLINERS

Tools Required: -N/A

Tools Recommended: -N/A

Components Supplied: -IV POLE

 CAUTION: This part was designed for a specific repair, based on the build of the chair from the serial number you provided. If the part is used on another chair, you may get unexpected results. Discard these directions once the repair is complete.

Directions:

Please read all the directions before beginning the repair. Refer back to the directions as you make the replacement. Call if you have any questions or problems. Suggested tools are not required. Note that references to “left” or “right” are determined from the seated position.

 CAUTION: This chair has moving parts that may create pinch points. Work or cleaning should be performed only in the upright position with the chair latched or in the full recline position. Keep your hands clear of the mechanism when moving the chair from one position to another.

1. Assemble your tools and move the chair to your chosen work area.
2. Lock the casters, a carpeted work area is suggested.
3. Remove the existing IV pole (IVPOLE1) by lifting it up and out of the mounting bracket, and discard
4. Verify that the roll pin is at the bottom of the IV pole bracket, and orient the replacement IV pole upright (IVPOLE1).
5. Slide the replacement IV pole down into the IV pole bracket.
6. Verify proper installation, the IV pole and bracket should be mounted securely.