


QQ-30506
CW-CUSHION WEDGE

Product: - CHAMPION RECLINERS

Tools Required: -N/A

Tools Recommended: -N/A

Components Supplied: -CUSHION WEDGE

 CAUTION: This part was designed for a specific repair, based on the build of the chair from the serial number you provided. If the part is used on another chair, you may get unexpected results. Discard these directions once the repair is complete.

Directions:

Please read all the directions before beginning the repair. Refer back to the directions as you make the replacement. Call if you have any questions or problems. Suggested tools are not required. Note that references to “left” or “right” are determined from the seated position.

 CAUTION: This chair has moving parts that may create pinch points. Work or cleaning should be performed only in the upright position with the chair latched or in the full recline position. Keep your hands clear of the mechanism when moving the chair from one position to another.

1. Remove existing cushion wedge and discard (if applicable).
2. Place the cushion wedge zipper-side down on the seat of the chair alongside the length of arm. The widest part of the cushion wedge should be oriented towards the back.
3. The cushion wedge may be repositioned from one side of the chair to the other for patient comfort.