


QQ-30509
300 SERIES FOLEY HOOK

Product:	-310 & 31X RECLINERS
Tools Required:	-#8 SQUARE BIT -IMPACT GUN
Tools Recommended:	-N/A
Components Supplied:	-FOLEY HOOK -PROVIDED HARDWARE

 **CAUTION:** This part was designed for a specific repair, based on the build of the chair from the serial number you provided. If the part is used on another chair, you may get unexpected results. Discard these directions once the repair is complete.

Directions:

Please read all the directions before beginning the repair. Refer back to the directions as you make the replacement. Call if you have any questions or problems. Suggested tools are not required. Note that references to “left” or “right” are determined from the seated position.

 **CAUTION:** This chair has moving parts that may create pinch points. Work or cleaning should be performed only in the upright position with the chair latched or in the full recline position. Keep your hands clear of the mechanism when moving the chair from one position to another.

1. Assemble your tools and move the chair to your chosen work area.
2. Lock the casters, a carpeted work area is suggested.
3. Identify the foley-hook to be replaced on either side of the chair.
4. Use a #8 square bit and impact gun to remove the 2 wood screws (SC-81WD) securing the foley-hook to the arm.
5. Discard the foley-hook and hardware removed in step 4.
6. Place replacement foley-hook (002117) in the same location, verify that the hook is in the upright position.
7. Use a #8 square bit and impact gun to secure the foley-hook to the arm using 2 replacement wood screws (SC-81WD).
8. Verify the foley hook is secure and functional— the replacement unit should not be loose.
9. Repeat steps 3-7 for the other side if applicable.