

300 Series, Alo Rec Mech Spring Set

CUSTOMER INSTRUCTIONS

For Models:

300, 310, 31X, 61xL, 62xL, 63xL, 64xL

Applicable for Service Kits:

201851K, 201763K (QQ-01010), 201610K (QQ-30004), 201611K (QQ-30005)

PLEASE READ AND FAMILIARIZE YOURSELF WITH ALL INSTRUCTIONS BEFORE USING THIS PRODUCT. DESIGN, APPEARANCE, PARTS, AND CONSTRUCTION ARE SUBJECT TO CHANGE WITHOUT NOTICE.

If you have trouble understanding these instructions, contact Customer Service before attempting to use this product; otherwise injury may occur.



TO REDUCE THE RISK OF INJURY TO PERSONS:

1. READ AND FOLLOW ALL DIRECTIONS.
2. This kit was designed for a specific repair, retrofit, or installation based on the serial number provided. If kit is used on a different model, unexpected results may occur.
3. This chair has moving parts that may create pinch points. Work or cleaning should be performed only in the upright position with the chair latched OR in the full recline position. Keep hands clear of the mechanism, and all other pinch points, when moving the chair from one position to another.
4. Immediately remove any chair with mechanical or visible damage from service.
5. USE ONLY AUTHORIZED REPLACEMENT PARTS.

Symbols



WARNING

General Warning - If the warning is not heeded, it can cause serious injury



CAUTION

Caution - If the precaution is not taken, it may cause minor or moderate injury

IMPORTANT

Important information

IMPORTANT

Left & Right is determined as if sitting in the chair.

Parts Included

Part Number	Description	Quantity
N/A	Outside Mechanism Springs - 1 Set	1 Set

Tools Needed

N/A


Procedure:

1. Lock the casters on the Recliner (**Central Locking Caster** model shown in images).



2. Open the mechanism and recline the chair completely.

Note: If the chair has the power option, use the hand pendant to recline the chair completely.

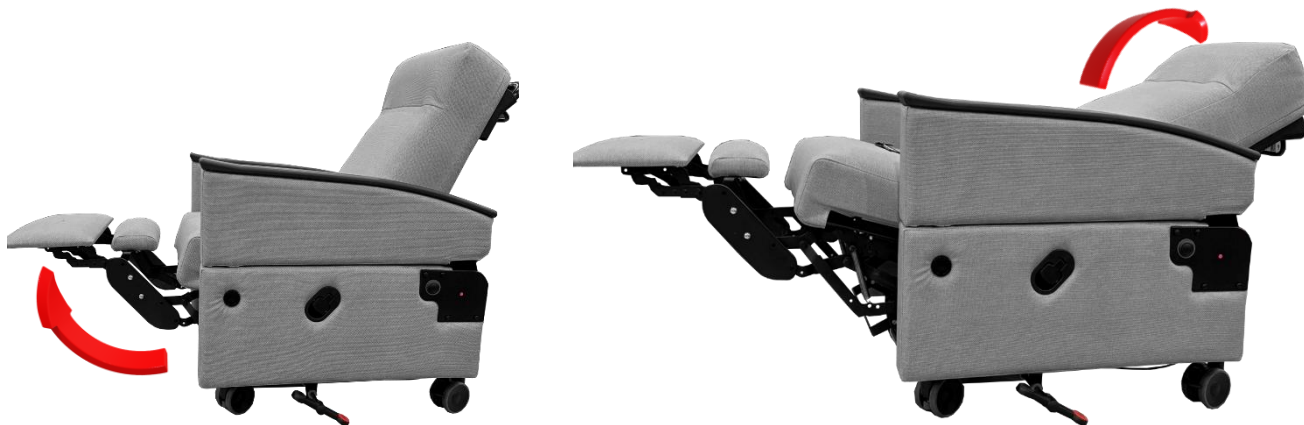

3. Locate the springs under the front of the seat as shown.



4. Carefully grasp the upper part of the spring to remove the spring hook from the recline mechanism as shown.



5. Unhook the bottom of the spring from the recline mechanism as shown. Discard the spring.



6. Install the new spring into the bottom of the spring hook location on the mechanism as shown. Hook the other end of the spring on the upper spring location on the recline mechanism as shown.



7. Repeat on the opposite side.

8. Raise the back of the chair and then close the chair completely.

Note: if the chair has the power option, use the hand pendant to close the chair completely.

9. Actuate the mechanism through all positions to ensure functionality.

