

QQ-30015
300 SERIES STD PUSH HANDLE

Product:	-300, 310 RECLINERS
Tools Required:	-#1 PHILLIPS SCREWDRIVER -#2 PHILLIPS SCREWDRIVER
Tools Recommended:	-#2 PHILLIPS BIT -IMPACT GUN
Components Supplied:	-PUSH HANDLE-STANDARD -INSTALLATION HARDWARE

 **CAUTION:** This part was designed for a specific repair, based on the build of the chair from the serial number you provided. If the part is used on another chair, you may get unexpected results. Discard these directions once the repair is complete.

Directions:

Please read all the directions before beginning the repair. Refer back to the directions as you make the replacement. Call if you have any questions or problems. Suggested tools are not required. Note that references to “left” or “right” are determined from the seated position.

 **CAUTION:** This chair has moving parts that may create pinch points. Work or cleaning should be performed only in the upright position with the chair latched or in the full recline position. Keep your hands clear of the mechanism when moving the chair from one position to another.

1. Assemble your tools and move the chair to your chosen work area.
2. Lock the casters, a carpeted work area is suggested.
3. Locate the push handle (001663) that is being replaced.
4. Near the push handle, locate the trend covers on the left-hand and right-hand sides (002121 & 002122).
5. Use a #1 Phillips screwdriver to remove the 2 screws (SC-619516PPHZ) securing each trend cover to the trend brackets (DO NOT DISCARD HARDWARE). Set trend covers and hardware aside.
6. Use a #2 Phillips screwdriver to remove the 2 screws (SC-14134B) securing the push handle.
7. Discard the old push handle and removed hardware.
8. Align the replacement push handle (001663) to the back-frame mounting holes. Insert 2 screws (SC-14134B) and tighten with a #2 Phillips screwdriver (or recommended tools).
9. Re-install covers and hardware set aside in step 5. Use a #1 Phillips screwdriver and 2 screws (SC-619516PPHZ) to secure each trend cover to the trend brackets.
10. Verify the push handle is assembled correctly and that the chair is functional.