


QQ-30012
300 SERIES TREND COVERS

Product:	-300 SERIES RECLINERS
Tools Required:	-#1 PHILLIPS SCREWDRIVER
Tools Recommended:	-N/A
Components Supplied:	-TREND COVERS -INSTALLATION HARDWARE

 CAUTION: This part was designed for a specific repair, based on the build of the chair from the serial number you provided. If the part is used on another chair, you may get unexpected results. Discard these directions once the repair is complete.

Directions:

Please read all the directions before beginning the repair. Refer back to the directions as you make the replacement. Call if you have any questions or problems. Suggested tools are not required. Note that references to “left” or “right” are determined from the seated position.

 CAUTION: This chair has moving parts that may create pinch points. Work or cleaning should be performed only in the upright position with the chair latched or in the full recline position. Keep your hands clear of the mechanism when moving the chair from one position to another.

1. Assemble your tools and move the chair to your chosen work area.
2. Lock the casters, a carpeted work area is suggested.
3. Near the push handle, locate the trend covers on the left-hand and right-hand sides (002121 & 002122).
4. Use a #1 Phillips screwdriver to remove the 2 screws (SC-619516PPHZ) securing each trend cover to the trend brackets. Discard the trend covers and removed hardware.
5. Install replacement trend covers (002121 & 002122) and hardware. Use a #1 Phillips screwdriver and 2 screws (SC-619516PPHZ) to secure each trend cover to the trend brackets.
6. Verify that the chair is functional.