Michael Graves with Stryker Overbed Table  
Table-pont Michael Graves de Stryker

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# Symbols and Definitions

## SYMBOLS

	Warning/Caution: Consult accompanying documentation
	Safe Working Load
	Pinch Point
	Manufacturer

## WARNING/CAUTION/NOTE DEFINITION

The words **WARNING**, **CAUTION** and **NOTE** carry special meanings and should be carefully reviewed.

### **WARNING**

Alerts the reader about a situation which, if not avoided, could result in death or serious injury. It may also describe potential serious adverse reactions and safety hazards.

### **CAUTION**

Alerts the reader of a potentially hazardous situation which, if not avoided, may result in minor or moderate injury to the user or patient or damage to the equipment or other property. This includes special care necessary for the safe and effective use of the device and the care necessary to avoid damage to a device that may occur as a result of use or misuse.

### **NOTE**

Provides special information to make maintenance easier or important instructions clearer.

# Introduction

This manual is designed to assist you with the operation of Stryker Model 4810 Overbed Table. Read this manual thoroughly before using the equipment or beginning maintenance on it. To ensure safe operation of this equipment, it is recommended that methods and procedures be established for educating and training staff on the safe operation of this unit.

## INTENDED USE

The Model 4810 Michael Graves with Stryker Overbed Table is a wheeled table with an adjustable height surface. This table is to be used for the temporary storage of patient food and personal items. The table is not to be used for medical procedures.

## PRODUCT DESCRIPTION

The Stryker Model 4810 Overbed Table is designed for ease of use in mobility and cleaning. A large, welcoming surface provides space for personal items, while obvious touch-points allow users to easily position the table to their comfort.

## SPECIFICATIONS

	Safe Working Load*	20 lb	9 kg
Overall Length / Width		36.5 in / 21.25 in	92.5 cm / 54 cm
Weight		45 lb	20.5 kg
Top Surface Height Minimum / Maximum		27.75 in / 43.75 in	70.5 cm / 111 cm

\* **Note:** Loading beyond the safe working load could cause damage to the overbed table.

**Stryker reserves the right to change specifications without notice.**

# Introduction

---

English


## CONTACT INFORMATION

Contact Stryker Customer Service or Technical Support at: (800) 327-0770 or (269) 324-6500.

Stryker Medical  
3800 E. Centre Avenue  
Portage, MI 49002  
USA

Please have the serial number (A) of your Stryker product available when calling Stryker Customer Service or Technical Support. Include the serial number in all written communication.

## SERIAL NUMBER LOCATION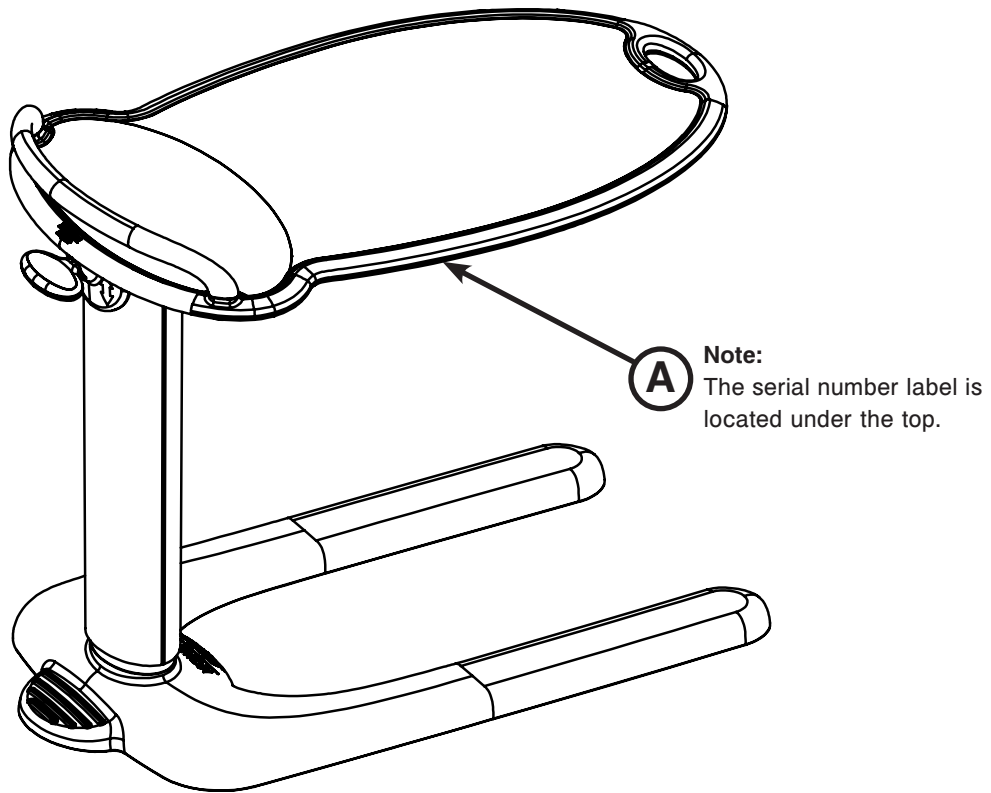


# Summary of Safety Precautions

## WARNING

- Use caution when raising and lowering the overbed table to avoid pinching your fingers between the activation lever and release cover.
- Do not lean on table. Leaning on table may cause the table to tip which may result in injury to the individual or damage to the unit.
- Do not sit on table. Sitting on table may cause the table to tip which may result in injury to the individual or damage to the unit.
- To avoid the risk of personal injury or damage to the table, support the table top before removing the screws that secure it to the upper weldment.
- To avoid the risk of personal injury or damage to the table, use caution when lifting the top assembly up and out of the lower column as the top and weldment are heavy and awkward.

## CAUTION

- Failure to properly clean a contaminated surface will increase the risk of exposure to bloodborne pathogens.
- Do not apply lubricant of any kind to the gas spring. Lubricant will damage the gas spring.

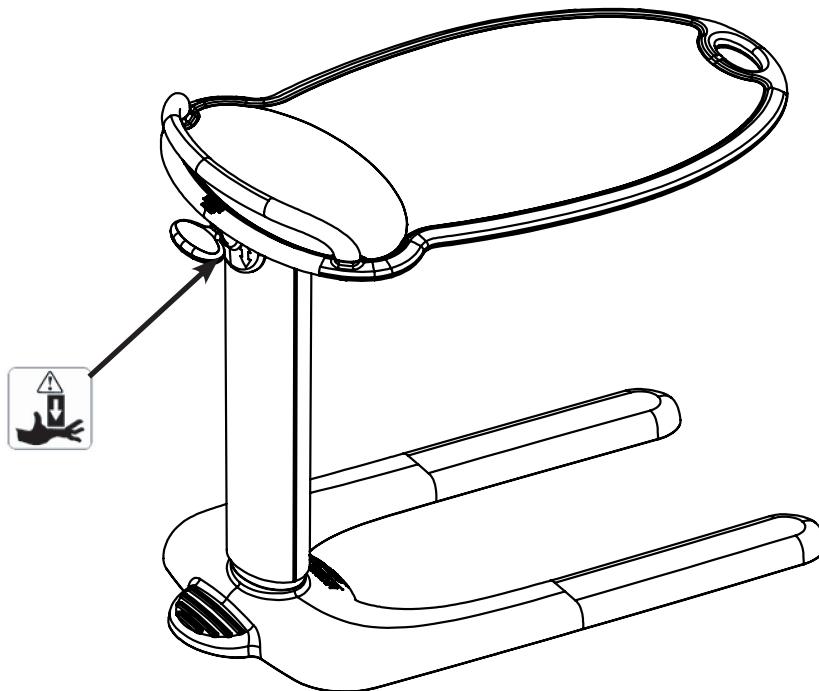# Operation Guide

English

## RAISING THE OVERBED TABLE

To raise the overbed table, either:

- Lift up on the underside of the activation lever as shown in Figure 1 or
- Lift up on the provided handle as shown in Figure 2.

### **WARNING**

Use caution when raising and lowering the overbed table to avoid pinching your fingers between the activation lever and release cover.

Figure 1: Lift Up on Activation Lever

Figure 2: Lift Up on Handle

## LOWERING THE OVERBED TABLE

To lower the overbed table, push down on the activation lever as shown in Figure 3.

### **WARNING**

- Do not lean on table. Leaning on table may cause the table to tip which may result in injury to the individual or damage to the unit.
- Do not sit on table. Sitting on table may cause the table to tip which may result in injury to the individual or damage to the unit.

Figure 3: Push Down on the Activation Lever

## MOVING THE OVERBED TABLE

Use the push bar (Figure 4) or integrated handle (Figure 5) to move the table.

**Figure 4: Push Bar**

**Figure 5: Integrated Handle**

When encountering obstacles, hold the push bar and push down on the kick plate to raise the front casters as shown in Figure 6.

**Figure 6: Hold Push Bar and Push Down on Kick Plate**

# Preventative Maintenance

English

Preventative maintenance should be performed at a minimum of annually. A preventative maintenance program should be established for all Stryker Medical equipment. Preventative maintenance may need to be performed more frequently based on the usage level of the product.

### CHECKLIST

- All fasteners secure
- Top and laminate intact - not damaged
- Top edge molding intact - not damaged
- Release handle locking and releasing properly
- All casters secure
- No debris in casters and they roll/swivel freely
- Base hood intact - not damaged

 **CAUTION**

Do not apply lubricant of any kind to the gas spring. Lubricant will damage the gas spring.

Unit Serial Number:		

Completed by: \_\_\_\_\_ Date: \_\_\_\_\_

## Cleaning

A recommended cleaner is a chlorinated bleach solution (5.25% - less than 1 part bleach to 100 parts water). Thoroughly rinse the solution from the product surface. If other hospital-grade cleaning agents are used, follow the instructions provided by the cleaning solution's manufacturer. Do not allow cleaning solutions or other fluids to pool on the unit. Wipe the surface with a soft, dry cloth to remove any moisture.

 **CAUTION**

Failure to properly clean a contaminated surface will increase the risk of exposure to bloodborne pathogens.

# Quick Reference Replacement Parts List

---

The parts and accessories listed on this page are all currently available for purchase. Some of the parts identified on the assembly drawing parts in this manual may not be individually available for purchase. Please call Stryker Customer Service USA at 1-800-327-0770 for availability and pricing.

<b>Part Name</b>	<b>Part Number</b>
Activation Lever	4810-200-025
Caster Wheel	3150-001-200
Gas Spring Assembly	3150-002-310
Hospitality Handle	4810-200-035
Top, Windsor/Silver	4810-210-021
Top, Maple/Silver	4810-210-022
Top, White/Iron	4810-210-023
Top, Pear/Silver	4810-210-024

# Troubleshooting Guide

English

Identify the problem from the table below and follow the instructions for resolution. If assistance is needed at any time during troubleshooting, please contact a service technician at 1-800-327-0770.

Problem / Failure	Recommended Action
Table drifts up.	Make sure that the original table top remains attached to the base. If so, then replace the gas spring (see <a href="#">page 1-15</a> ).
Table drifts down without weight on table top.	If the table is difficult to raise and it drops back down when released, replace the gas spring (see <a href="#">page 1-15</a> ).
Table has a spongy feeling.	This is normal as long as it does not drift down.
Table drifts down with weight on table top.	Adjust the gas spring lock pin for more clearance, so that there is a little play in the handle.
Table randomly drifts down with light weight load applied.	Replace the gas spring (see <a href="#">page 1-15</a> ).

## TOP REPLACEMENT

### Tools Required:

- T27 Torx Driver
- T25 Torx Driver
- #2 Phillips Screwdriver

---

### WARNING

To avoid the risk of personal injury or damage to the table, support the table top before removing the screws that secure it to the upper weldment.

---

### Procedure:

1. Raise the unit to the highest position.
2. Using a T27 Torx driver, remove the two screws that hold the hospitality handle to the table top.
3. Using a T25 Torx driver, remove the three screws that hold the upper column cover to the top support weldment.
4. Gently slide the column cover down (including the release lever cover).
5. Using a #2 Phillips screwdriver, remove the six screws that secure the top weldment.
6. Reverse steps to reinstall.
7. Verify proper operation of the unit before returning it to service.

## ACTIVATION LEVER REPLACEMENT

### Tools Required:

- T25 Torx Driver
- 7/16" Combination Wrench
- 5/32" Allen Wrench

### Procedure:

1. Using a T25 Torx driver, remove the three screws that secure the upper column cover to the top support weldment.
2. Gently slide the column cover down (including the release lever cover).
3. Using a 5/32" Allen wrench and a 7/16" combination wrench, remove the nut and bolt that secure the activation lever to the top support weldment.
4. Discard the activation lever and remove the existing spring to reuse.
5. Reverse steps to reinstall.
6. Verify proper operation of the unit before returning it to service.

## UPPER COLUMN REPLACEMENT

### Tools Required:

- T25 Torx Driver
- T27 Torx Driver

### Procedure:

1. Using a T25 Torx driver, remove the three T25 Torx screws from the top of the upper column and lower the column down to the base.
2. Using a T27 Torx driver, remove the two opposing T27 Torx screws that connect the upper bushing to the lower column.
3. Lift the top assembly straight up and out of the lower column and set it aside.

---

### **WARNING**

To avoid the risk of personal injury or damage to the table, use caution when lifting the top assembly up and out of the lower column as the top and weldment are heavy and awkward.

---

4. Reverse steps to reinstall.
5. Verify proper operation of the unit before returning it to service.

## GAS SPRING REPLACEMENT

### Tools Required:

- T25 Torx Driver
- T27 Torx Driver
- 5/32" Allen Wrench
- 7/16" Combination Wrench
- Pliers


### Procedure:

1. Using a T25 Torx driver, remove the three T25 Torx screws from the top of the upper column.
2. Lower the column down to the base.
3. Using a 5/32" Allen wrench and a 7/16" combination wrench, remove the shoulder bolt and nut that secure the activation lever to the column to remove the lever.
4. Using pliers, remove the gas spring release lever (as shown) from the gas cylinder. Gently push down on the lever and pull it out while moving it slightly from side to side.
5. Using a T27 Torx driver, remove the two opposing T27 Torx screws that connect the upper bushing to the lower column.
6. Lift the top assembly out of the lower column and set it aside.
7. Remove the rue clip from the clevis pin that holds the gas spring assembly to the top weldment and remove the clevis pin.
8. Reverse steps to reinstall.
9. Verify proper operation of the unit before returning it to service.

### Notes:

When reinstalling the gas cylinder, align the threaded stud with the hole in the bottom of the lower column before placing the top assembly above the lower column.

When reinstalling the upper bushing, compress the gas spring slightly before tightening the two T27 Torx screws that connect the upper bushing to the lower column.

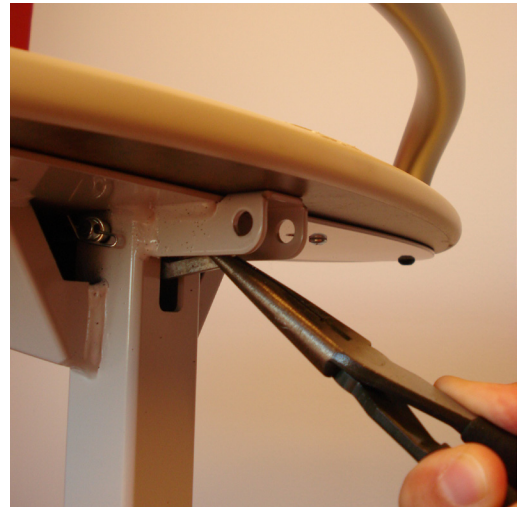Gas Spring Release Lever Removal

# Service Information

English

## CASTER ASSEMBLY REPLACEMENT

### Tools Required:

- Cordless Drill, 3/16" Drill Bit
- Pop Rivet Gun with 3/16" Tip

### Procedure:

1. Fully raise the table top.
2. While maintaining a secure grip on the table, carefully flip it forward or turn it on its side to allow access to the damaged caster.
3. Using a cordless drill and 3/16" drill bit, drill out the two rivets and remove the damaged caster.
4. Using a pop rivet gun with a 3/16" tip, install the new caster using two new pop rivets.

## CASTER REPLACEMENT

### Tools Required:

- Wheel removal tool (3150-001-202)

### Procedure:

1. Fully raise the table top.
2. While maintaining a secure grip on the table, carefully tip it forward or turn it on its side to allow access to the damaged caster.
3. Using the supplied wheel removal tool (3150-001-202), remove the caster wheel.

**NOTE:** Make sure that the wheel is facing the wheel removal tool with the stem facing away from the wheel removal tool.

4. Press to install the new caster wheel (3150-001-201).
5. Verify proper operation of the product before returning it to service.

## BASE COVER REPLACEMENT

### Tools Required:

- #2 Phillips Screwdriver
- T25 Torx Driver
- T27 Torx Driver

---

### WARNING

To avoid the risk of personal injury or damage to the table, support the table top before removing the screws that secure it to the upper weldment.

---


### Procedure:

1. While holding the upper column, use a T25 Torx driver and remove the three T25 Torx screws from the top of the upper column then lower the column down to the base.
2. Using the T27 Torx driver, remove the two opposing T27 Torx screws that connect the upper bushing to the lower column.
3. Lift the top assembly straight up and out of the lower column and set it aside.
4. Lift the upper column off the lower column and remove it from the base.  
**NOTE: It is easier to perform the next step while sitting on a chair.**
5. Flip the base and lower column over and rest the lower column on the top of your thigh.
6. Using the T27 Torx driver, remove the four T27 Torx screws from the bottom of the base that hold the column assembly and remove the lower column.
7. Turn the base and cover over. Using the Phillips screwdriver, remove the six #8 Phillips screws that hold the cover to the base.
8. Turn the base back over and remove the cover.
9. Reverse steps to reinstall.
10. Verify proper operation of the unit before returning it to service.

# Overbed Table Assembly

English

4810-010-000 Rev C (Reference Only)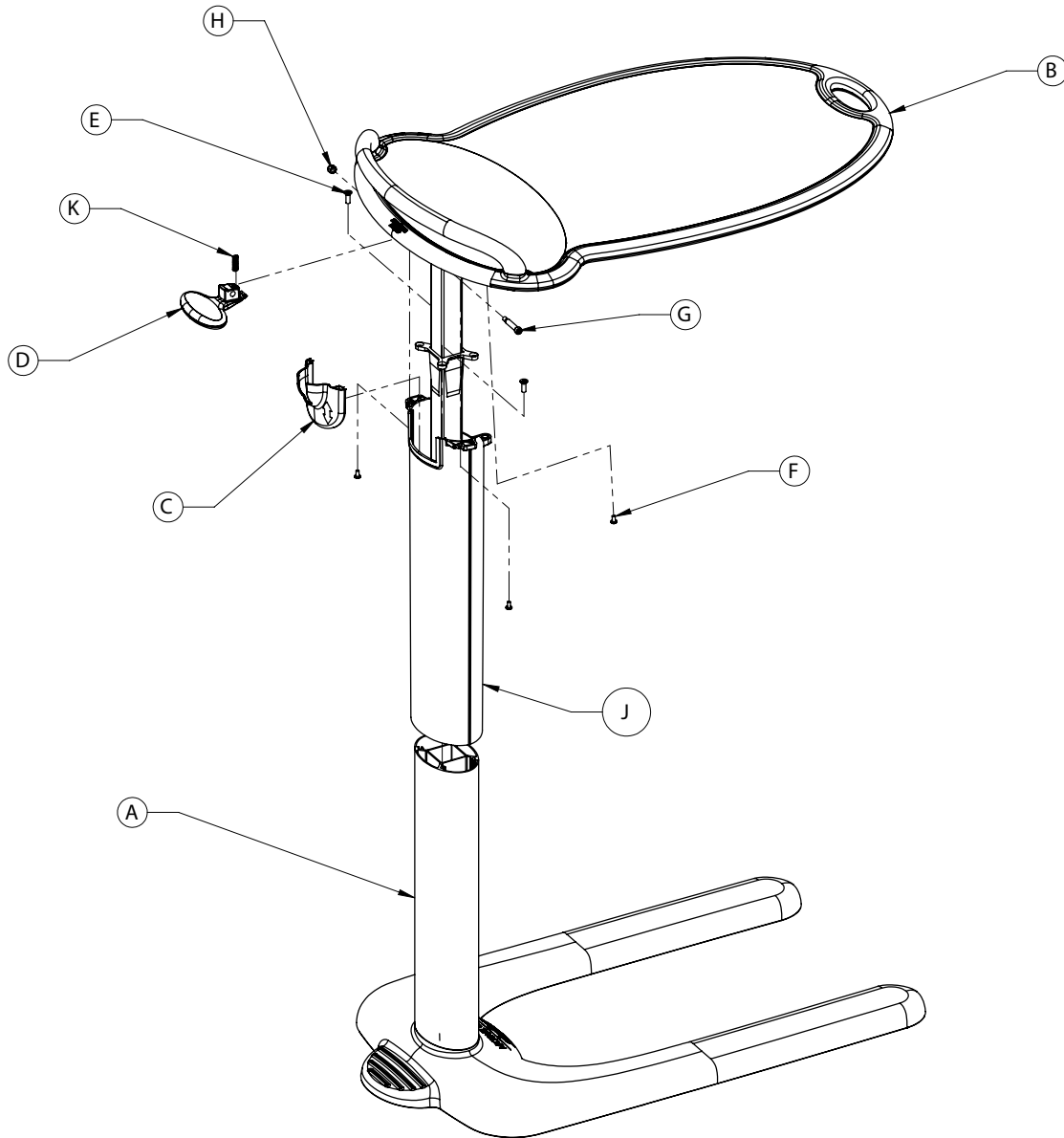


Item	Description	Qty.	4810-010-001 Windsor/Silver	4810-010-002 Maple/Silver	4810-010-003 White/Iron	4810-010-004 Pear/Silver
A	Base Assembly ( <a href="#">page 1-19</a> )	1	4810-110-002*	4810-110-003*	4810-110-003*	4810-110-002*
B	Top Assembly ( <a href="#">page 1-20</a> )	1	4810-210-011*	4810-210-012*	4810-210-013*	4810-210-014*
C	Release Cover	1	4810-100-021	4810-100-024	4810-100-024	4810-100-021
D	Activation Lever	1	4810-200-025	4810-200-025	4810-200-025	4810-200-025
E	Flat Head Screw	2	0001-163-000	0001-163-000	0001-163-000	0001-163-000
F	Pan Head Machine Screw	3	0004-136-000	0004-136-000	0004-136-000	0004-136-000
G	Shoulder Bolt	1	0008-086-000	0008-086-000	0008-086-000	0008-086-000
H	Fiberlock Hex Nut	1	0016-028-000	0016-028-000	0016-028-000	0016-028-000
J	Upper Column	1	3150-002-290	3150-002-294	3150-002-294	3150-002-290
K	Compression Spring	1	0038-604-000	0038-604-000	0038-604-000	0038-604-000

\* Shown for reference only. These assemblies are not available for sale.  
[Return To Table of Contents](#)

# Base Assembly

4810-110-000 Rev A (Reference Only)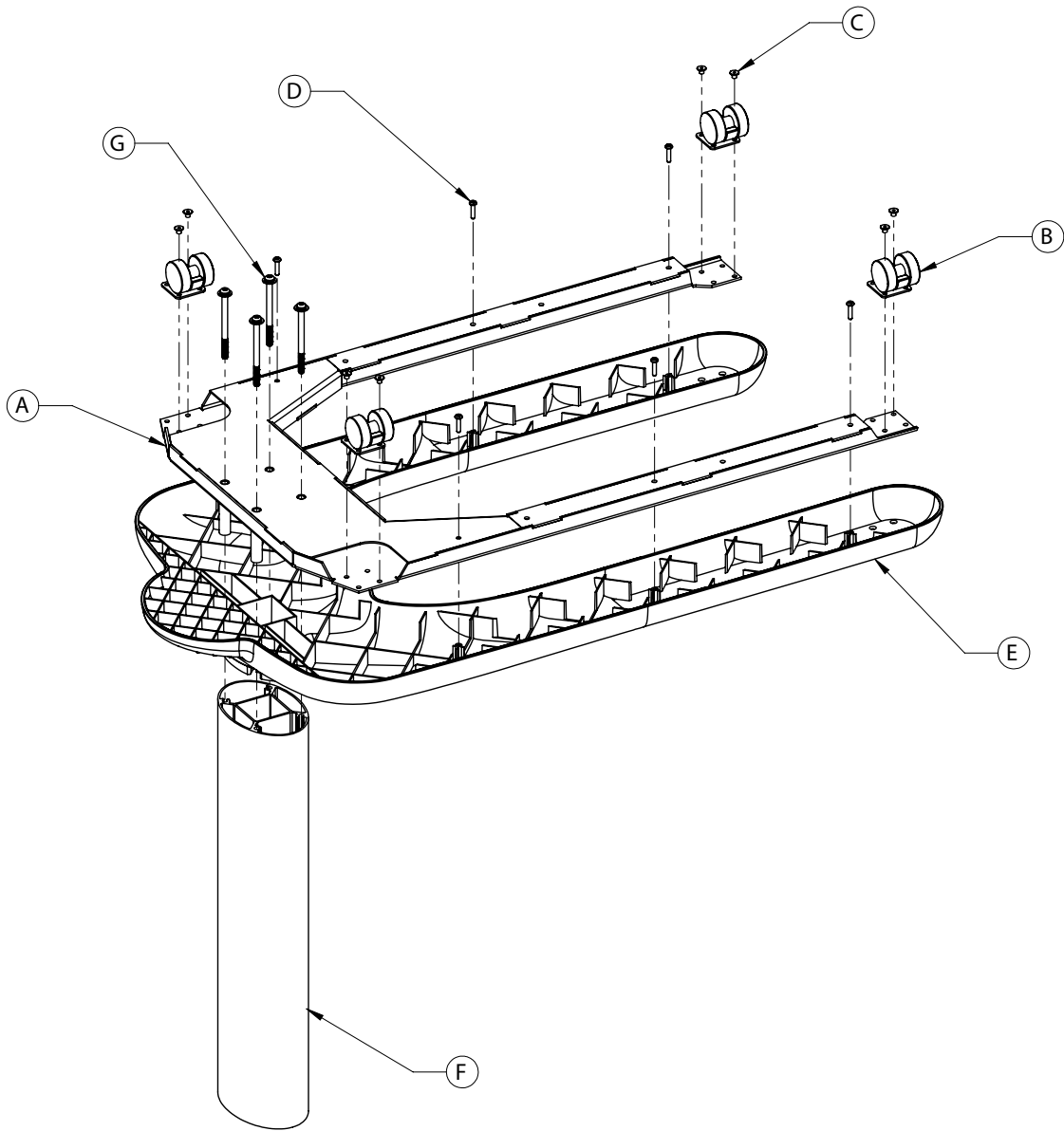


Item	Description	Qty.	4810-110-002 Dark Gray	4810-110-003 Stone
A	Base Weldment	1	3150-001-105	3150-001-105
B	Caster	4	3150-001-200	3150-001-200
C	Rivet	8	0025-122-000	0025-122-000
D	Phillips Pan Head Screw	6	0023-116-000	0023-116-000
E	Base Cover	1	4810-100-012	4810-100-013
F	Lower Column	1	3150-002-100	3150-002-100
G	Torx Flat Head Screw	4	3150-001-223	3150-001-223

[Return To Table of Contents](#)

# Top Assembly

English

4810-210-010 Rev A (Reference Only)

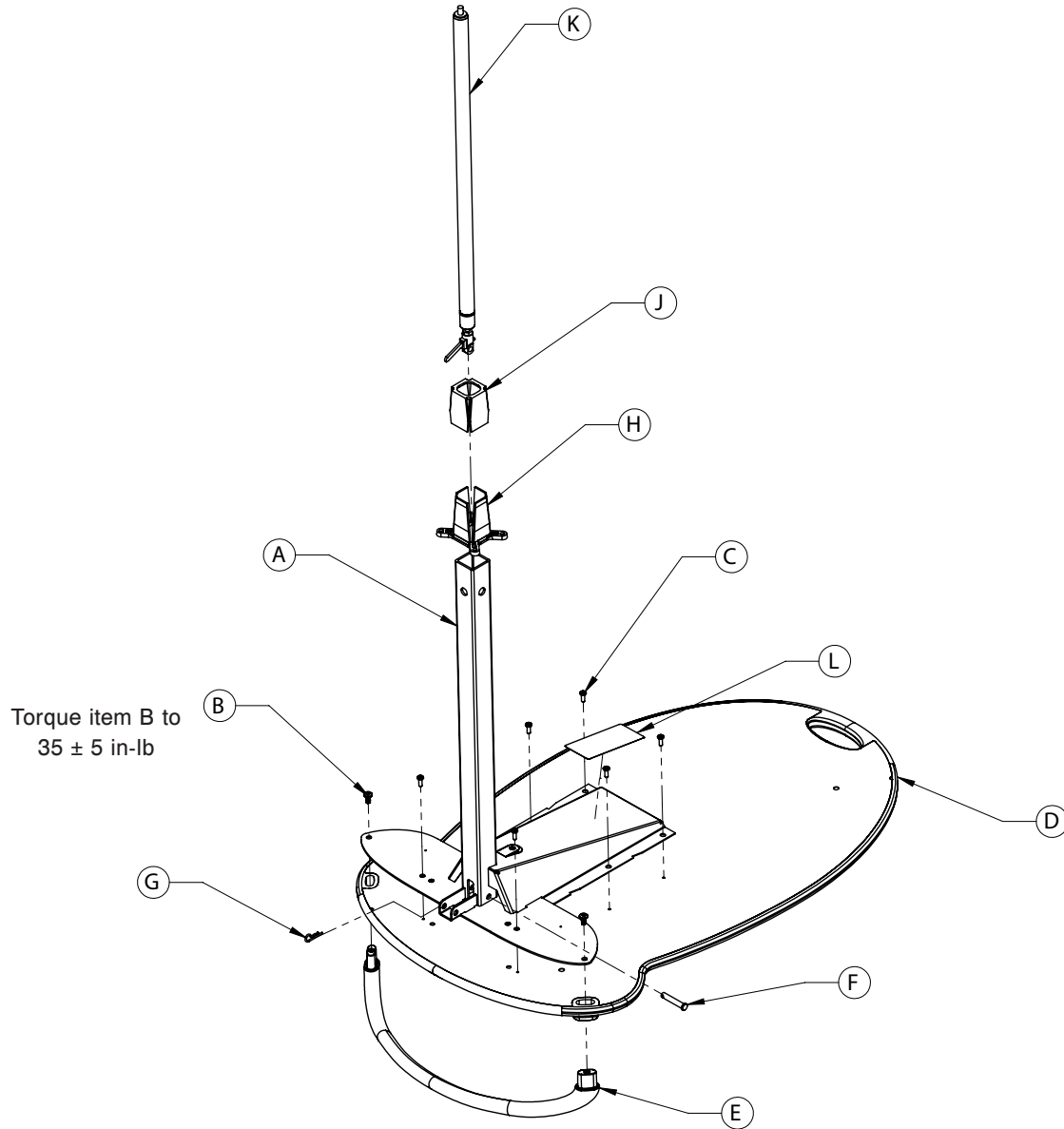Item	Description	Qty.	4810-210-011 Windsor/Silver	4810-210-012 Maple/Silver	4810-210-013 White/Iron	4810-210-014 Pear/Silver
A	Top Weldment	1	4810-210-046	4810-210-046	4810-210-046	4810-210-046
B	Button Head Cap Screw	2	0004-589-000	0004-589-000	0004-589-000	0004-589-000
C	Phillips Pan Head Screw	6	0023-256-000	0023-256-000	0023-256-000	0023-256-000
D	Top	1	4810-210-021	4810-210-022	4810-210-023	4810-210-024
E	Hospitality Handle	1	4810-200-035	4810-200-035	4810-200-035	4810-200-035
F	Gas Spring Clevis	1	0026-296-000	0026-296-000	0026-296-000	0026-296-000
G	Rue Clip	1	0027-019-000	0027-019-000	0027-019-000	0027-019-000
H	Bushing, Upper	1	3150-002-410	3150-002-410	3150-002-410	3150-002-410
J	Bushing, Lower	1	3150-002-405	3150-002-405	3150-002-405	3150-002-405
K	Gas Spring Assembly	1	3150-002-310	3150-002-310	3150-002-310	3150-002-310
L	Label	1	3150-009-100	3150-009-100	3150-009-100	3150-009-100

[Return To Table of Contents](#)

# Warranty

---

Stryker Medical stands behind the craftsmanship of our products. Providing our customers quality products is our mission, and we pledge to do everything we can to quickly resolve any problems encountered within the terms of the warranties listed below.

## THREE YEAR WARRANTY

Stryker warrants the Michael Graves with Stryker Overbed Table for three (3) years under normal use and proper care. For three years from date of shipment, Stryker will repair or, at our option, replace defective merchandise free of charge (parts, labor and shipping included). **Note:** One (1) year warranty on the gas spring assembly.

The warranties listed above are void in cases of modifications, misuse, improper assembly or maintenance, accidents, damage caused by a carrier other than Stryker Medical.

Michael Graves with Stryker products are designed for a 7 year expected service life under normal use, conditions, and with appropriate periodic maintenance as described in the maintenance manual for each device.

Stryker makes no other warranty or representation, either expressed or implied, except as set forth herein. There is no warranty of merchantability and there are no warranties of fitness for any particular purpose. In no event shall Stryker be liable hereunder for incidental or consequential damages arising from or in any manner related to sales or use of any such equipment.

## TO OBTAIN PARTS AND SERVICE

Stryker products are supported by a nationwide network of dedicated Stryker Field Service Representatives. These representatives are factory trained, available locally, and carry a substantial spare parts inventory to minimize repair time. Simply call your local representative, or call Stryker Customer Service USA at 1-800-327-0770.

## RETURN AUTHORIZATION

Merchandise cannot be returned without approval from the Stryker Customer Service Department. An authorization number will be provided which must be printed on the returned merchandise. Stryker reserves the right to charge shipping and restocking fees on returned items. **Special, modified, or discontinued, items not subject to return.**

## DAMAGED MERCHANDISE

ICC Regulations require that claims for damaged merchandise must be made with the carrier within fifteen (15) days of receipt of merchandise. **Do not accept damaged shipments unless such damage is noted on the delivery receipt at the time of receipt.** Upon prompt notification, Stryker will file a freight claim with the appropriate carrier for damages incurred. Claim will be limited in amount to the actual replacement cost. In the event that this information is not received by Stryker within the fifteen (15) day period following the delivery of the merchandise, or the damage was not noted on the delivery receipt at the time of receipt, the customer will be responsible for payment of the original invoice in full. Claims for any short shipment must be made within thirty (30) days of invoice.

## INTERNATIONAL WARRANTY CLAUSE

This warranty reflects U.S. domestic policy. Warranty outside the U.S. may vary by country. Please contact your local Stryker Medical representative for additional information.

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