

# Symmetry<sup>®</sup> Sleeper Bench

Operations Manual

REF 3500














# Table of Contents

- Symbols .....2
- Introduction .....3
  - Summary of safety precautions .....3
  - Product description .....3
  - Expected service life .....3
  - Specifications .....3
  - Contact information .....4
  - Serial number location for reference only .....4
  - Cleaning .....4
  - Service parts .....5
- Warranty .....8
  - Warranty exclusion and damage limitations .....8
  - To obtain parts and service .....8
  - Return authorization .....8
  - Damaged product .....8
  - International warranty clause .....8

# Symbols

	Operating instructions/Consult instructions for use
	General warning
	Caution
	Catalogue number
	Serial number
	Manufacturer
	Safe working load
	Distributed by
	<a href="http://www.stryker.com/patents">www.stryker.com/patents</a>

# Introduction

This manual assists you with the operation or maintenance of your Stryker product. Read this manual before operating or maintaining this product. Set methods and procedures to educate and train your staff on the safe operation or maintenance of this product.

## Note

- This manual is a permanent part of the product and should remain with the product even if the product is sold.
- Stryker continually seeks advancements in product design and quality. This manual contains the most current product information available at the time of printing. There may be minor discrepancies between your product and this manual. If you have any questions, contact Stryker Customer Service or Technical Support at 1-800-327-0770.

## Summary of safety precautions

Always read and strictly follow the warnings and cautions listed on this page. Service only by qualified personnel.

---

### WARNING

- Tipping or collapsing can result in serious injury.
  - Safe working load 350 lb (159 kg).
  - Frequently inspect the bench for visible cracks, and tighten all screws and bolts.
  - Failure to follow these warnings could result in serious injury.
- 

### CAUTION

- Improper usage of the product can cause injury to the guest or operator. Operate the product only as described in this manual.
  - Do not modify the product or any components of the product. Modifying the product can cause unpredictable operation resulting in injury to guest or operator. Modifying the product also voids its warranty.
- 


## Product description

The Stryker model 3500 **Symmetry™** Sleeper Bench is low height seating that saves space. The bench provides guest seating and a sleeping surface for overnight guests.

## Expected service life

The **Symmetry** Sleeper Bench has a 10 year expected service life under normal use conditions and with appropriate periodic maintenance.

## Specifications

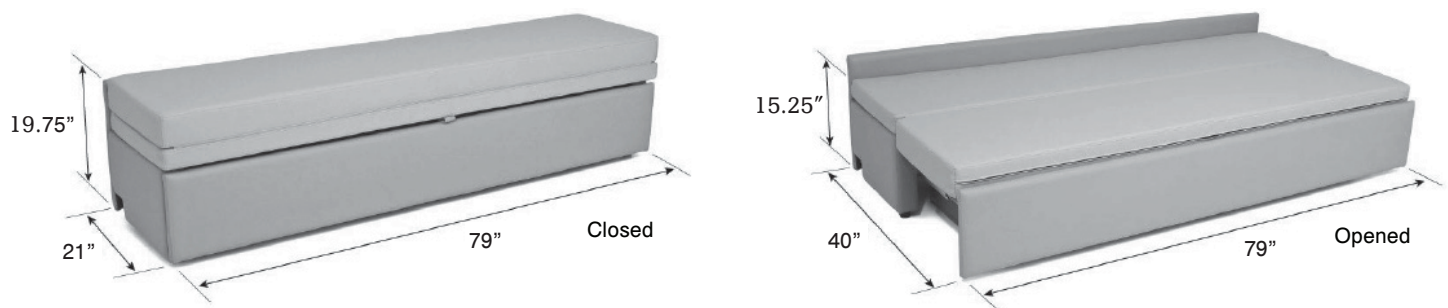

Model	3500-000-670
Safe working load	350 lb (159 kg)
Depth of bench (closed)	21" (53.34 cm)
Depth of bench (open)	40" (101.6 cm)
Width of bench	79" (200.66 cm)
Height of bench	19.75" (50.16 cm)
Height of bed (opened)	15.25" (38.73 cm)
Seat	Plywood
Seat foam	Ultracell
Seat foam thickness	7.0" (17.8 cm)
Product weight	141 lb (63.9 kg)
Casters	3" rear casters
Fabric	Customize fabric and finishes

## Contact information


Contact Stryker Customer Service or Technical Support at: 1-800-327-0770.

Stryker Medical  
3800 E. Centre Avenue  
Portage, MI 49002  
USA

To view your operations or maintenance manual online, see <https://techweb.stryker.com/>.

Have the serial number (A) of your Stryker product available when calling Stryker Customer Service or Technical Support. Include the serial number in all written communication.


## Serial number location for reference only

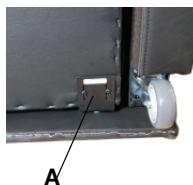


## Cleaning

1. Hand wash the product with warm water and a mild soap solution.
2. Rinse off soap residue.
3. Dry the product.

# Service parts

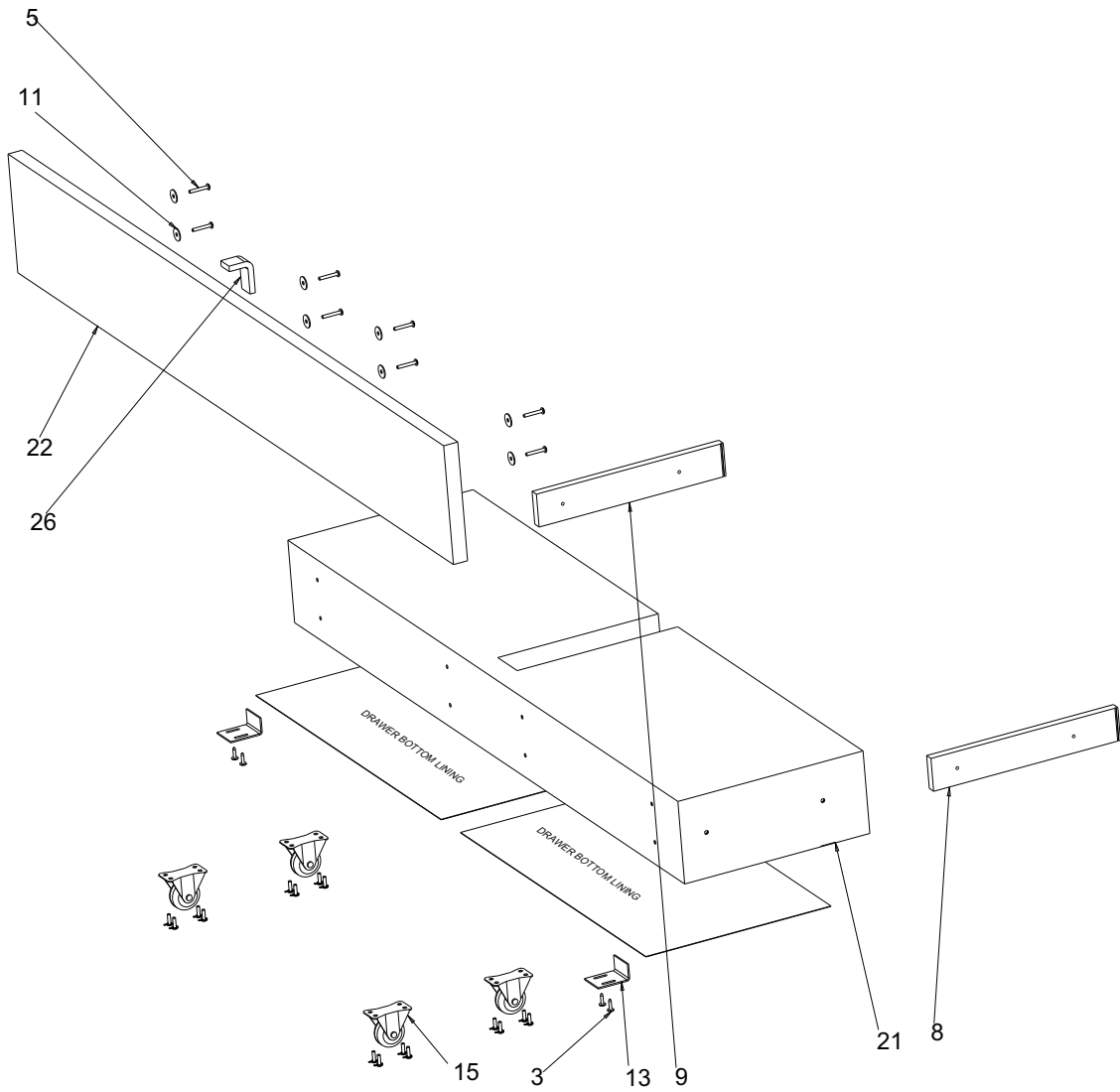

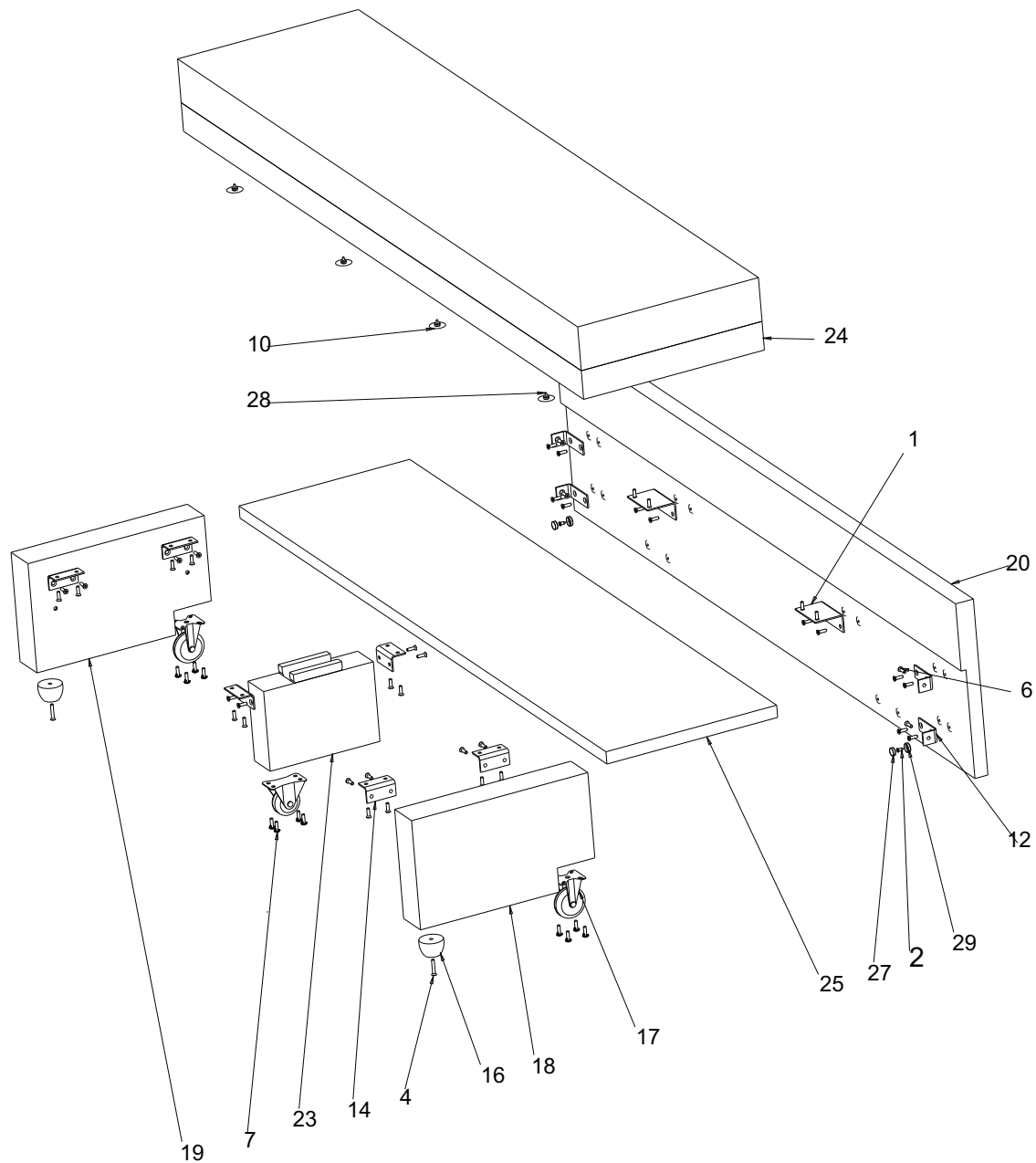

Item	Number	Name	Quantity
1	3500-100-205	4560 bracket	2
2	3500-100-206	Flat socket zinc	2
3	3500-100-207	Pan quad screw ts blk	4
4	3500-100-208	Flat quad ms blk	2
5	3500-100-082	Truss quad blk	8
6	3500-100-210	Flat quad blk	44
7	3500-100-079	Truss quad serrated screw	28
8	3500-100-212	MTL drawer slide, lh	1
9	3500-100-213	MTL drawer slide, rh	1



Item	Number	Name	Quantity
10	3500-100-214	3/8 BS x 1-1/4 od x 1/16th	4
11	3500-100-069	5/16"IDS 1-1/8"ODX .0747" thk	8
12	3500-100-216	Bracket long	4
13	3500-100-217	Bracket magnet	2
14	3500-100-218	Bracket short	6
15	3500-100-219	Drawer caster	5
16	3500-100-220	Plastic glide black	2
17	3500-100-221	Side arm caster	2
18	3500-100-222	UPH arm support, lh	1
19	3500-100-223	UPH arm support, rh	1
20	3500-100-224	UPH back board	1
22	3500-100-225	UPH front rail	1
23	3500-100-226	UPH middle support arm	1
24	3500-700-227	Kit, seat cushion assembly	<ul style="list-style-type: none"> <li>• (1) UPH seat cushion</li> <li>• (4) 3/8 BS x 1-1/4OD x 1/16th</li> <li>• (4) Snap stud with screw V/N 1102</li> </ul>
25	3500-100-228	UPH seat deck	1
26	3500-100-229	Handle strip	1
27	3500-100-230	Magnet	2
28	3500-100-231	Snap stud	4
29	3500-100-232	Steel cup for magnet	2

## Warranty

Stryker Medical warrants our Symmetry® Bench Sleeper seating for the lifetime of the product, classified as 10 years, under normal use and proper care. Subject to the limitations set below, Stryker will repair or, at our option, replace defective merchandise free of charge.

- 10 years on wood
- 5 years on mechanism
- 3 years on casters, mattress and Stryker Medical textiles
- 3 years on foam

## Warranty exclusion and damage limitations

The express warranty set forth herein is the only warranty applicable to the product. **Any and all other warranties, whether express or implied, including any implied warranty of merchantability or fitness for a particular purpose are expressly excluded by Stryker.** In no event shall Stryker be liable for incidental or consequential damages.

## To obtain parts and service

Stryker products are supported by a nationwide network of dedicated Stryker Field Service Representatives. These representatives are factory trained, available locally, and carry a substantial spare parts inventory to minimize repair time. Simply call your local representative or call Stryker Customer Service at 1-800-327-0770.

## Return authorization

Product cannot be returned without prior approval from the Stryker Customer Service Department. An authorization number will be provided which must be printed on the returned product. Stryker reserves the right to charge shipping and restocking fees on returned product. Special, modified, or discontinued products are not subject to return.

## Damaged product

ICC Regulations require that claims for damaged product must be made within fifteen (15) days of receipt of the product. Do not accept damaged shipments unless such damage is noted on the delivery receipt at the time of receipt. Upon prompt notification, Stryker will file a freight claim with the appropriate carrier for damages incurred. Claims will be limited in amount to the actual replacement cost. In the event that this information is not received by Stryker within the fifteen (15) day period following the delivery of the product, or the damage was not noted on the delivery receipt at the time of receipt, the customer will be responsible for payment of the original invoice in full within thirty (30) days of receipt. Claims for any incomplete shipments must be made within thirty (30) days of invoice.

## International warranty clause

This warranty reflects U.S. domestic policy. Warranty outside the U.S. may vary by country. Contact your local Stryker Medical representative for additional information.



# stryker<sup>®</sup>



Stryker Medical  
3800 East Centre Ave.  
Portage, MI 49002  
USA