SofCare®

HeelCare® Boot, Botte, Stövel

| | |
|-----|--------------|
| REF | 2760-100-000 |
| REF | 2760-200-000 |
| REF | 2760-300-000 |

Operations Manual
Manuel d'utilisation
Manual de Utilização
Användarhandbok

ENGLISH (EN)
FRANÇAIS (FR)
PORTUGUÊS (PT)
SVENSKA (SV)

2016/03 C.0 2760-309-001 REV C www.stryker.com

Introduction

This manual assists you with the operation of the Stryker Model 2760-100-000, 2760-200-000, 2760-300-000 **SofCare® HeelCare®** Boot. Read this manual before operating this product and keep a copy on file. Set methods and procedures to educate and train your staff on the safe operation of this product.

Intended use

The **SofCare HeelCare** Boot assists in the prevention and treatment of pressure ulcers for all categories or stages. The categories or stages include stages I, II, III, IV, unstageable, and deep tissue injury. This single patient use product is for use after clinical assessment of the lower extremities associated with extremity edema; extent, pattern and distribution. Edema (swelling), a localized or generalized abnormal accumulation of fluid in the tissues, may obscure the appearance of normal anatomy and may impact the circulatory status of the lower extremities.

Product description

An inflated **SofCare HeelCare** Boot is a pre-inflated, air filled device with adjustable straps that secure the boot to the patient's leg. The boot reduces the pressure on the patient's heel.

Expected life

The **SofCare HeelCare** Boot has a 30 day single patient use expected life under normal use conditions.

Contraindications

None known.

Specifications

| Model | 2760-100-000 | 2760-200-000 | 2760-300-000 |
|--------------------------|-------------------------|---------------|--------------|
| Material | Poly Nonwoven | Poly Nonwoven | Nylon |
| Calf dimensions | 6 - 19 in. (15 - 48 cm) | | |
| Max weight of lower calf | 31.5 lb (14.29 kg) | | |

| Operating ambient temperature range and humidity limits | Storage temperature and humidity limits |
|---|---|
| | |

Symbols

| | | | |
|--|-------------------|--|---|
| | General warning | | Caution |
| | Batch code | | Operating instructions |
| | Safe working load | | Model number |
| | No sharp objects | | No steam or high temperatures |
| | Manufacturer | | Do not walk on product |
| | CE mark | | Authorized representative in the European Community |

Stryker reserves the right to change specifications without notice.

Contact information

Contact Stryker Customer Service or Technical Support at 1-800-327-0770.

Stryker Medical
3800 E. Centre Avenue
Portage, MI 49002 USA

Have the Lot batch code available when you call Customer Service. Include the Lot batch code in all written communication.

Operation

WARNING

- Do not reuse this product to avoid the risk of cross-contamination and infection.
- Always check the patient's skin at regular intervals for redness or breaks. If redness or breaks in the skin occurs, consult the attending physician.
- Always inspect the product for tears, cuts, holes, or any other damage, before use.

CAUTION

Do not clean models 2760-100-000 and 2760-200-000. If soiled, dispose of the product according to hospital protocol. This is a single patient use product.

Applying the product to the patient

- Measure the calf to select the appropriate product size (Figure 1).
- Insert the patient's foot so the heel protrudes through the hole in the boot (Figure 2).
- Apply the toe strap loose to make sure the product fits (see Figure 2).
- Slide a finger along both sides of the of the foot by the toes (Figure 3).

Notes:

- Iodine based disinfectants will stain the product. Staining does not affect the patient nor the use of the product.
- Check the patient at regular intervals as directed by hospital protocol.

Cleaning (Model 2760-300-000 only)

- Clean the product with a clean damp cloth, soap, and water.
- Wipe dry.
- Dispose of product according to your local waste management policy.

Note: Disinfectant contact will degrade the product. Do not use disinfectants, such as solvents, alcohol based products, or Virex TB germicidal products.

Warranty

Stryker Medical, a division of Stryker Corporation ("Stryker") warrants that its model 2760 **SofCare HeelCare** Boot product will be free from defects in manufacturing and workmanship for a maximum of 30 day use period. "Use" period begins when such devices individual package is opened. Stryker's obligation under this warranty is expressly limited to supplying a product replacement, at its option, any product which is, in the sole discretion of Stryker, found to be defective. If requested by Stryker, products for which a claim is made shall be returned prepaid to the factory. Any improper use or any alterations or repair by others in such manner as in Stryker's judgement affects the product materially and adversely shall void this warranty. Any repair of Stryker products using parts not provided or authorized by Stryker shall void this warranty. No employee or representative of Stryker is authorized to change this warranty in any way.

Warranty exclusion and damage limitations

The express warranty set forth herein is the only warranty applicable to the product. **Any and all other warranties, whether express or implied, including any implied warranty of merchantability or fitness for a particular purpose are expressly excluded by Stryker.** In no event shall Stryker be liable for incidental or consequential damages.

Return authorization

Product cannot be returned without prior approval from the Stryker Customer Service Department. An authorization number will be provided which must be printed on the returned product. Stryker reserves the right to charge shipping and restocking fees on returned product. Special, modified, or discontinued products are not subject to return.

Damaged merchandise

ICC Regulations require that claims for damaged product must be made within fifteen (15) days of receipt of the product. Do not accept damaged shipments unless such damage is noted on the delivery receipt at the time of receipt. Upon prompt notification, Stryker will file a freight claim with the appropriate carrier for damages incurred. Claims will be limited in amount to the actual replacement cost. In the event that this information is not received by Stryker within the fifteen (15) day period following the delivery of the product, or the damage was not noted on the delivery receipt at the time of receipt, the customer will be responsible for payment of the original invoice in full within thirty (30) days of receipt. Claims for any incomplete shipments must be made within thirty (30) days of invoice.

International warranty clause

This warranty reflects U.S. domestic policy. Warranty outside the U.S. may vary by country. Please contact your local Stryker Medical representative for extra information.

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