# Instructions For Use Sof-Care® Mattress Series

Read and understand these instructions and precautions before beginning any work.

If you have any questions or problems, please contact your local dealer or local sales representative.

## 1.0 Indications for use

This device is intended to assist in treatment and prevention of pressure ulcers.

## 2.0 Contraindication

Air support therapy is not recommended when spinal stability is a concern.

## 3.0 Safety Precautions

Review the following SAFETY PRECAUTIONS prior to using the Sof-Care Mattress Series.

### WARNING

- Explosion risk. Do not use in the presence of flammable anesthetics.
- Electrical shock risk. Refer all servicing to qualified personnel.
- Disconnecting the power cord will remove the AC power supply to the mattress and discontinue therapy.
- Disinfect the Sof-Care Mattress Series between patient installations. Failure to disinfect may risk cross-contamination and infection.
- Check patient at least every 8 hours or, once per shift, to assure proper system inflation. Otherwise, desired therapy may not occur.
- Medical electrical equipment requires special precautions regarding EMC and needs to be installed and put into service according to the EMC information (provided within this Instructions For Use) to prevent equipment malfunction.
- Portable and mobile RF communication equipment can affect Medical Electrical Equipment.
- It is the caregiver's responsibility to monitor the patient's condition at regular intervals to ensure patient safety.
- Consult a physician if redness or skin breakdown occurs.

CAUTION: Some medical conditions may not respond to therapy of this type. Check patient's skin frequently. Consult physician if any redness or skin breakdown occurs. Serious injury could result if the patient's skin condition is left unchecked.

GAYMAR promotes the clinical assessment of each patient's condition and appropriate therapy administration by the caregiver.

### CAUTION

- The mattress cover must be completely dry before storage or adding linens. Failure to remove excess disinfectant could cause degradation of the cover material.
- Follow all applicable safety rules and protocols concerning patient and caregiver safety.
- Monitor the patient's skin condition regularly to ensure skin integrity.
- Patients with body weight or size near the recommended limits should be monitored more frequently to verify desired results are being achieved.
- Avoid contact of sharp objects with the mattress. Punctures, cuts and tears in the cover could result in contamination of the mattress, prevent proper air pressure control and compromise therapy and safety.

Type BF Equipment

Dangerous Voltage

Caution, consult accompanying documents

Consult instructions for use

Protective Earth (ground)


Temperature Limitation

Humidity Limitation

#### Care Instructions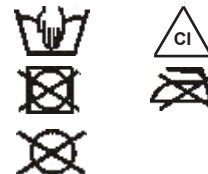

**Damp wipe only.**  
**10% chlorine bleach.**  
**Do not tumble dry.**  
**Do not iron.**  
**Do not dry-clean.**

NOTE: The inflated mattress has been tested for CPR procedures, but a backboard is recommended.

## Instructions For Use Sof•Care® Mattress Series


### 4.0 TECHNICAL DESCRIPTION

The Sof•Care Mattress series are reactive therapeutic mattresses that assist in the prevention and treatment of pressure ulcers. The mattress consists of a foam crib and an integrated pump which inflates a multi zoned air cell to provide pressure redistribution. The Sof•Care Mattress LAL and V-Mattress have the low air loss capability by connecting it to a Plexus Air Express pump (C1500). The foam crib assembly of the Sof•Care V-Mattress will allow for collapsing of the foot section of the mattress, when the bedframe is collapsed.

### 5.0 INSTRUCTIONS FOR USE:

1. If using a Sof•Care V-Mattress go to step 2, if not place mattress on frame with footend markings at proper end of bed, proceed to step 3.
2. a) Place Sof•Care V-Mattress on Hill-Rom Versacare bed frame with foot end markings at proper end of bed.
  - b) Standing at the foot end of the bed, lift the mattress up.
  - c) Locate the hold down bar on the bottom of the end of mattress.
  - d) Slide one side of the hold down bar into the metal loop.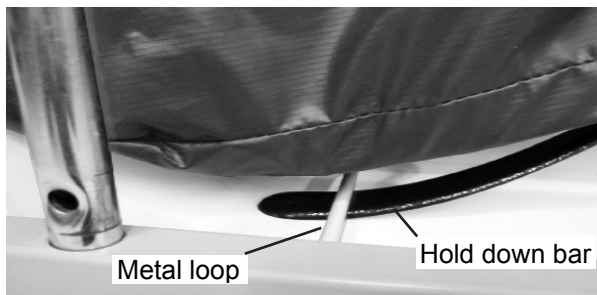


- e) Push the mattress all the way to the side.
- f) Set the mattress down and slide the mattress towards the opposite side, centering the mattress.

- g) Pull up on the mattress to assure that it is attached to the foot of the bed.

3. (Optional) Apply linens utilizing “D” rings for flat sheet. To secure linens (a), the Sof•Care Mattress series (b), thread four corners through D-rings (c), attached to mattress as shown in below. To ensure proper therapy, do not pull linens taut. Linens should remain loose and wrinkly on surface of the mattress to avoid hammocking.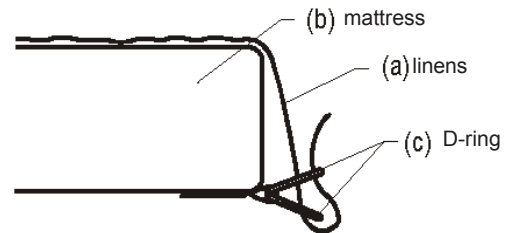

4. Plug the power cord into a properly grounded outlet with appropriate line voltage. Observe that the power plug illuminates and that the air filled support surface inflates. The seat section will inflate within 20 minutes.
5. The mattress will be fully inflated within 40 minutes.


FOR OPTIONAL LOW AIR LOSS ON SOF•CARE LAL AND SOF•CARE V MATTRESS: (Use Gaymar C1500 LAL Air Express Control Unit and Hose Assembly P/N 100727000)

1. Suspend control unit on footboard of bed using the mounting brackets on back of unit or rest unit on the floor.
2. Lift flap at end of bed and connect hose assembly quick connect to fitting on control unit and the other end of hose to the mattress.
3. Plug the AC power cord into properly grounded outlet and adjust the controller as needed for more or less low air loss.

**CAUTION:** Check patient's skin regularly. Consult physician if any redness or skin breakdown occurs. Serious injury could result if the patient's skin condition is left untreated.

## 6.0 TO DEFLATE THE MATTRESS

1. Unzip the mattress cover to expose the fire barrier.
2. Unfold the fire barrier material from underneath the crib assembly.
3. Slide the fire barrier to expose the pump assembly.  
NOTE: Do not remove the tywrap around the cord.
4. Slide the thumb latch on the barb fitting located on the pump assembly and then insert the male barb fitting (Gaymar P/N: 91367034). Repeat the process on the second barb fitting.  
NOTE: It is helpful to push on the cells to expel air more quickly.
5. Once deflated, slide the thumb latch on the barb fitting to remove the male barb fitting.

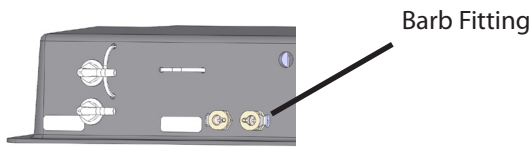6. Pull the fire barrier over the crib and pump assembly.
  - a) For Sof•Care V-Mattress, fold the material on the left side of the power cord under the mattress. Repeat the same on the right side.
  - b) For Sof•Care Mattress, fold the material under the mattress.
7. Zip the mattress cover closed.

## 7.0 TROUBLE SHOOTING CELLS:

1. Non-functioning cells can easily be spotted by looking for large depressions on the mattress surface.

Seat Section Cell Deflated

Head and Foot Cell Deflated

## 8.0 CLEANING AND CARE:

1. To clean, use soap and a clean damp cloth to wipe down the power cord and mattress. Do not use abrasive cleaners on the mattress. Wipe dry with a clean, dry cloth.  
Note: Blood and other body fluids must be thoroughly cleaned from all surfaces before applying disinfectants.
2. Apply disinfectant to the external surfaces of the control unit, hoses and mattress. Allow to completely dry. The solution contact time is what makes disinfection effective.
3. Wipe down the mattress with a clean, dry cloth to remove any excess disinfectant.

## 9.0 SPECIFICATIONS:

### PHYSICAL

	Dimensions	Weight
Sof•Care V-Mattress	35" X 84" x 7" 89 x 213 x 18 (cm)	28 lbs (13 kg)
Sof•Care Mattress	35" x 84" x 6 1/2" 89 x 213 x 17 (cm)	33 lbs (15 kg)
Sof•Care Mattress LAL	35" x 80" x 6 1/2" 89 x 203 x 17 (cm)	30 lbs (14 kg)

Max Patient Load	350 Lbs
Operating Ambient Temperature Range	60°F to 85°F (15°C to 29°C)
Relative Humidity	0% to 95%
Storage and Transport Temperature Range	-20°F to 120°F (-28°C to 48°C)
Relative Humidity	0% to 95%

## Instructions For Use Sof•Care® Mattress Series

### ELECTRICAL

Classification	Type BF with no applied patient parts, Class I, trans portable equipment suitable for continuous operation. Not classified for protection against harmful ingress of liquid. Not classified for use in the presence of flammable anesthetics.
Input	~120 VAC, 60 Hz, 9 W
Power Cord	4.6 m (15 ft), 18 AWG cord with lighted hospital grade plug
Fuses (Located inside)	Subminiature, 250V, very fast acting pump. not accessible 1/4 ampere to the operator)
Current Leakage	100 microamperes max.
CODE APPROVALS	UL60601-1, IEC60601-1-2; CAN/CSA 22.2 No. 601.1

### 10.0 WARRANTY:

The foam crib, covers and Air Cell are warrantied for a period of 5 years. The pump is warrantied for a period of 2 years.

Gaymar equipment and products are warranted against defects in material and workmanship under normal use and operation from the date of shipment from Gaymar for the time period indicated. During the warranty period, Gaymar will repair or replace at its sole option, free of charge, any defective part or products returned, prepaid to Gaymar Industries, Inc., with prior authorization. See Return Policy below.

### 11.0 RETURN POLICY:

1. All products returned to Gaymar require Return Goods (RG) numbers. Authorization to return goods and RG numbers are obtained by calling Gaymar Customer Service at 1.800.828.7341 or 1.716.662.2551.
2. When inquiry is made, please supply model and serial number, purchase order number, if merchandise was bought on contract, and reason for the return.
3. In general, deleted, damaged and outdated merchandise will not be accepted for credit. A minimum restocking charge of 20% will be assessed on return of current merchandise.
4. All returns are to be shipped prepaid to:  
  
Gaymar Industries, Inc.  
10 Centre Drive  
Orchard Park, NY 14127 USA
5. Dealer should request Returned Goods number direct from Gaymar, in advance of return. All items and conditions subject to change without notice.

Gaymar®  
Your patients. Our commitment.  
Manufactured by:  
GAYMAR INDUSTRIES, INC  
10 Centre Drive  
Orchard Park, NY 14127-2295 USA  
www.gaymar.com  
Tel USA: 800.828.7341  
716.662.2551  
Fax USA: 800.993.7890  
716.662.0748

SOF•CARE® MATTRESS SERIES  
ARE MADE IN THE USA

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