## Medi-Temp™

### **REF** D25000 Series Blood/Fluid Warming Set

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
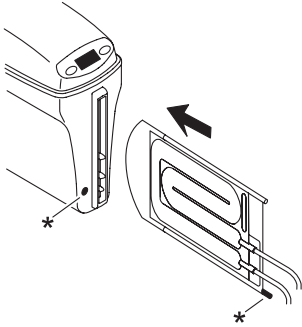
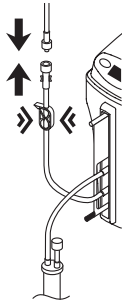
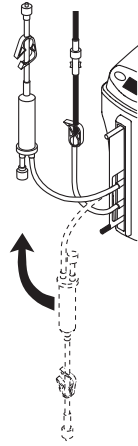
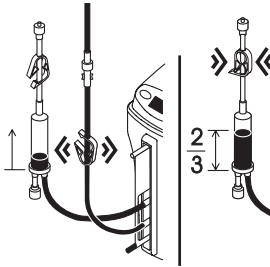
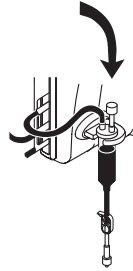
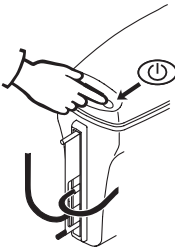
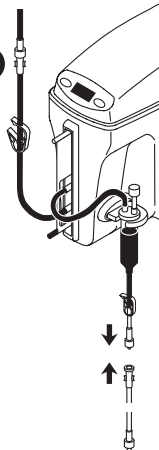
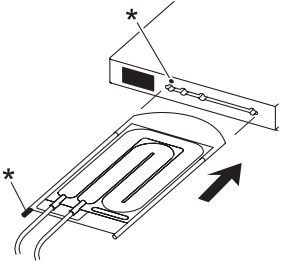
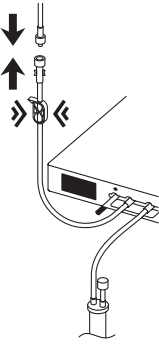
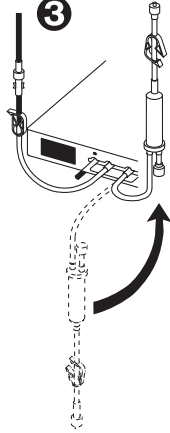
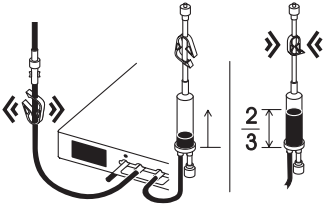
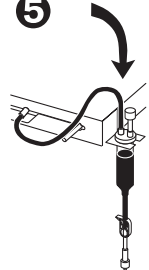
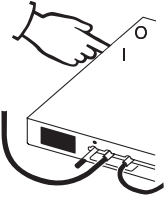
7

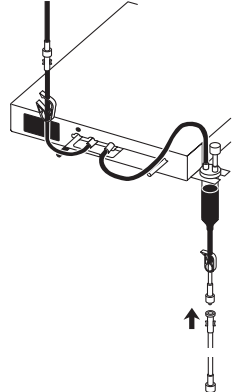8

All D2500CE Series Blood/Fluid Warming Sets are disposable, non-pyrogenic, sterile and for single use only.

To be used with FW300/FW400/FW600 Series Blood/Fluid Warmers only. Refer to *FW300/FW400/FW600 Series Blood/Fluid Warmer Operator's Manual* for complete operating procedures.

**Risk of cross-contamination (e.g. infection, allergic reaction) if product were to be reused.**

**Contains or Presence of Phthalates.**

**Instructions for Use - Setup**

Select a site for placement of the Warmer. If the Warmer is attached to a pole, ensure the pole clamp is securely tightened.

**CAUTION**

Do not attempt to transport the Warmer while mounted to an IV pole. It could tip over. Remove the Warmer for transport.

Plug the power cord into a properly grounded electrical outlet.

NOTE: Do not turn unit on without first inserting a Warming Set. Doing so may activate the Over Temperature Safety Thermostats. Also, do not prime the Warming Set prior to inserting it into the Warmer.

Follow the instructions below (see corresponding illustrations numbered 1 through 3 on pages 2 and 3):

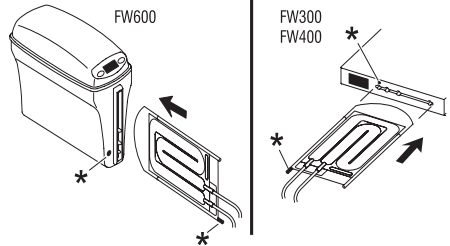
NOTE: Warming Sets are designed for use below 300 mm Hg.

- 1 Remove the Warming Set from its sterile packaging.

**WARNING**

Do not use the contents of the package if any part of the contents is damaged, distorted or contaminated. Contents are sterile and non-pyrogenic in unopened, undamaged package.

NOTE: The cassette is designed so it can be inserted into the Warmer in only one orientation. The guide rail with the blue end cap on the cassette lines up with the dot on the front of the Warmer (see figure below).

Insert the front end into the front of the Warmer, aligning the rails to each edge of the slotted opening.

Ensure that the cassette is completely inserted.

- ② Close Warming Set input clamp. Remove the protector from the female luer of the Warming Set and connect it to the male luer on the IV administration set.

Remove the protector from the male luer of the Warming Set. To minimize temperature drop and priming volume, remove unnecessary lead extensions at this time.

- ③ Invert the bubble trap.
- ④ Open Warming Set input clamp. Allow fluid to prime the Set.

When the bubble trap fills to approximately two-thirds (2/3) full, close output clamp.

- ⑤ Re-invert the bubble trap and mount it in the bubble trap holder so it is in a vertical position.

- ⑥ Turn unit on.
- ⑦ If required, connect an extension for patient connection.
- ⑧ Purge air from remaining length of tubing by opening output clamp and allowing fluid to flow.

**WARNING**

- All air must be removed from the fluid lines prior to connecting to the patient.
- Monitor the fluid lines to ensure they are air free. Never administer fluids if there are air bubbles in the line between the bubble trap and the patient connector.

**Failure to do so could result in the introduction of air to the patient, resulting in death or serious injury.**

**WARNING**

The bubble trap must be mounted and kept in the vertical position at all times. Failure to do so could result in the introduction of air to the patient. The introduction of air to the patient could result in death or serious injury.

D25000 Series Blood/Fluid Warmer Sets:

REF No.	Blood/Fluid Warmer Set
D25340CE	Standard Set
D25310CE	Standard Set with 48" Extensions (122 cm)
D25315CE	Standard Set with 60" Extensions (152 cm)
D25330CE	High Flow Set
D25310NL	Standard Set with 48" Extensions (122 cm) with Needleless Injection Port
D25320NL	Pediatric Set with Needleless Injection Port
D25330NL	High Flow Set with Needleless Injection Port

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Stryker France S.A.S.