

Birthing Bed

REF 4701

stryker®

Parts Manual



Table of Contents

4701-201-070.....	4
4701-201-075.....	5
3001-200-060.....	6
3001-200-050.....	7
3001-200-090.....	8
3001-200-080.....	9
4701-201-000.....	10
4701-101-020.....	17
4701-101-025.....	18
4701-001-030.....	19
4701-001-035.....	20
4712-032-100.....	21
4701-032-000.....	26
4712-032-015.....	31
4712-032-055.....	34
4701-032-075.....	35
4712-040-200.....	41
4701-040-000.....	42
4701-130-000.....	44
4701-130-110.....	60
5010-330-061.....	62
5010-330-060.....	63
4701-036-201.....	65
4701-140-225.....	66
4701-140-215.....	70
4701-140-220.....	74
4701-040-205.....	77
4712-035-050.....	78
4712-035-100.....	81
4701-035-000.....	85
4712-035-040.....	96
4712-035-045.....	98
4701-140-240/4701-140-245.....	99
4701-140-005.....	100
4701-040-025.....	106
4701-045-005/4701-045-015.....	108
4701-045-105/4701-045-110.....	109
4701-045-000/4701-045-010.....	110
4701-045-100.....	111
4712-048-105/4712-048-110.....	112
4701-048-000/4701-048-020.....	113
4712-048-100.....	114
4712-048-030.....	117

Table of Contents


4701-040-070.....	120
4701-040-060.....	123
4701-040-045/4701-040-055.....	126
4701-040-235.....	127
4701-040-130.....	128
4701-040-175.....	129
4701-040-180.....	130
4701-140-165.....	131
4701-140-170.....	133
4701-040-160.....	134
4701-040-185.....	136
4701-940-290.....	137
4701-940-270.....	138
4701-940-275.....	139
4701-940-295.....	140
4701-940-280/4701-940-285.....	141
4701-180-035.....	142
4701-180-040.....	143
4701-180-045.....	145
4701-108-055.....	146
4701-080-030.....	148
4701-080-000.....	151
4701-120-095.....	155
4701-120-085.....	156
4701-120-100.....	160
4701-120-090.....	161
3002-400-090.....	165
4701-120-065.....	166
4701-120-045.....	169
4701-120-070.....	172
4701-120-080.....	176
4701-120-060.....	179
4701-120-050.....	182
4701-120-075.....	186
4701-120-055.....	189
4701-120-005.....	192
4701-120-025.....	196
4701-120-010.....	199
4701-120-030.....	202
4701-120-020.....	205
4701-120-040.....	209
4701-120-015.....	212
4701-120-035.....	215

Table of Contents

5000-220-008.....	218
5000-220-010.....	219
5000-020-028.....	220
5000-020-029.....	221
5000-020-026.....	222
5000-020-027.....	223
5000-081-021/5000-081-023/5000-081-020.....	224
4701-130-125.....	226
4701-136-020.....	227
4701-136-010.....	228
4701-036-035.....	229
1211-211-010.....	230
1211-210-026.....	231
Recycling Passports	232


4701-201-070

Rev A

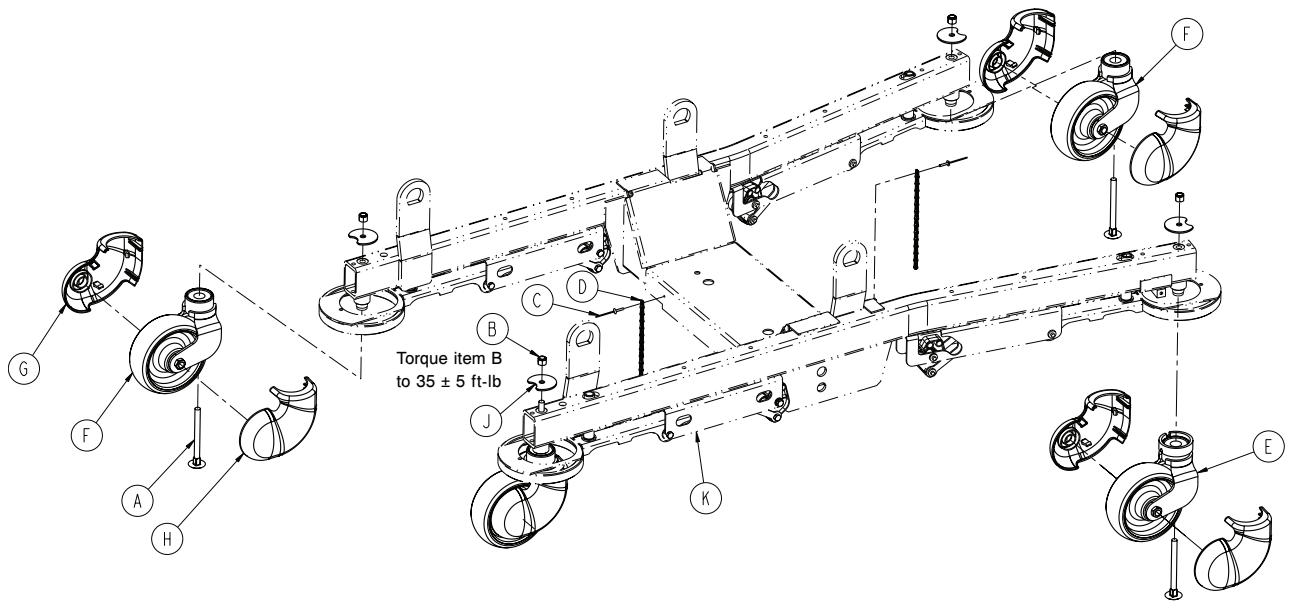

A	0005-042-000	4
B	0016-035-000	4
C	0025-103-000	2
D	0715-001-156	2
E	3001-200-050	1
F	3001-200-060	3
G	4701-001-095	4
H	4701-001-096	4
J	4701-001-094	4
K	4701-201-000	1


Rev A

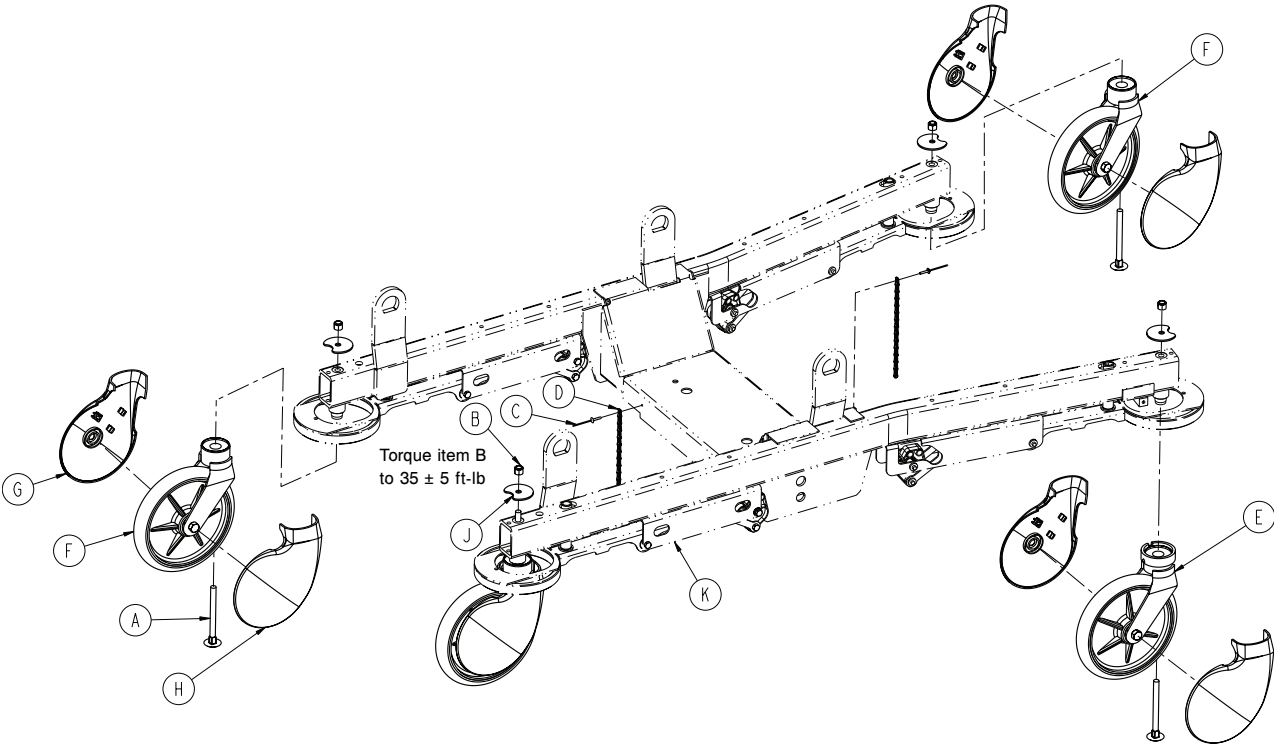

A	0005-042-000	4
B	0016-035-000	4
C	0025-103-000	2
D	0715-001-156	2
E	3001-200-080	1
F	3001-200-090	3
G	4701-001-077	4
H	4701-001-076	4
J	4701-001-094	4
K	4701-201-000	1


3001-200-060

Rev A

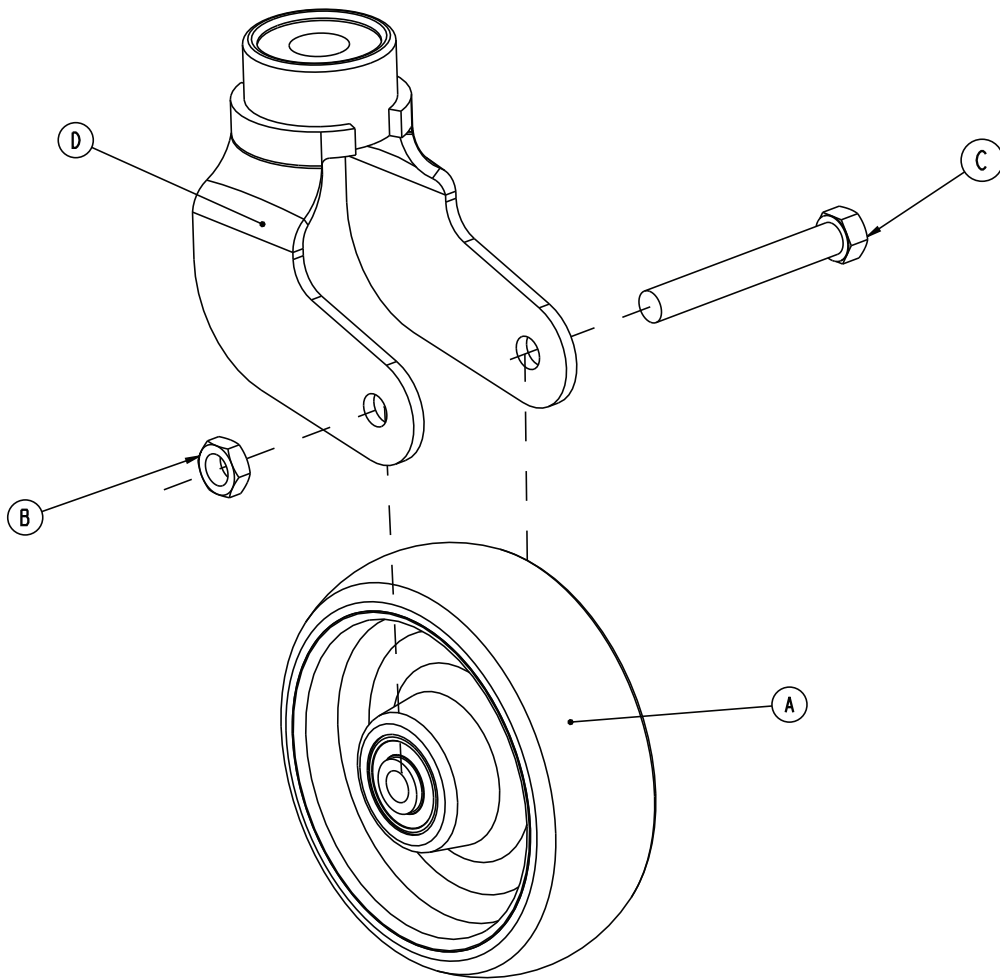


A	5000-002-010	1
B	0016-060-000	1
C	0003-342-000	1
D	3001-200-061	1

Rev -

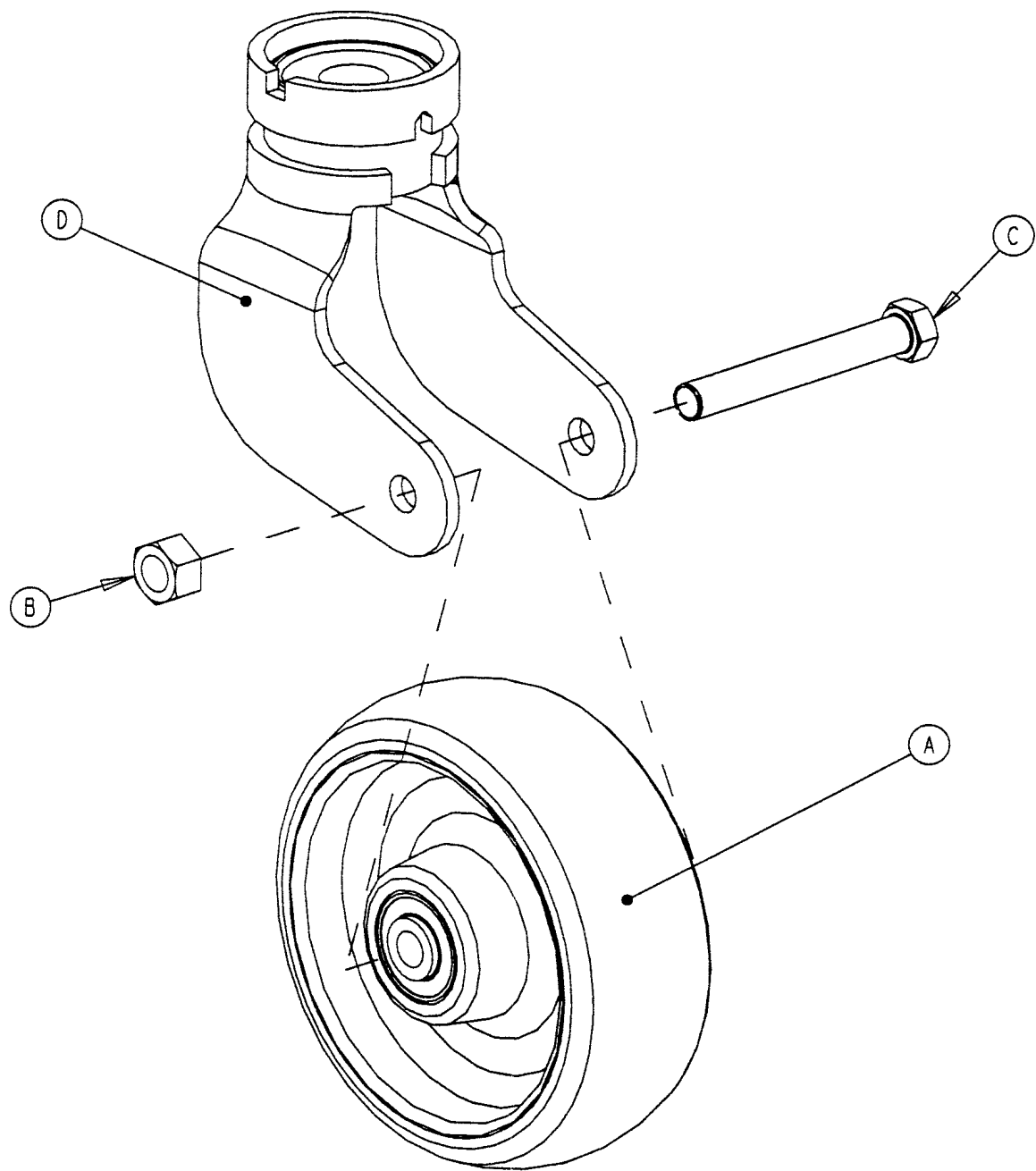


A	5000-002-010	1
B	0016-060-000	1
C	0003-342-000	1
D	3001-200-051	1

Rev A

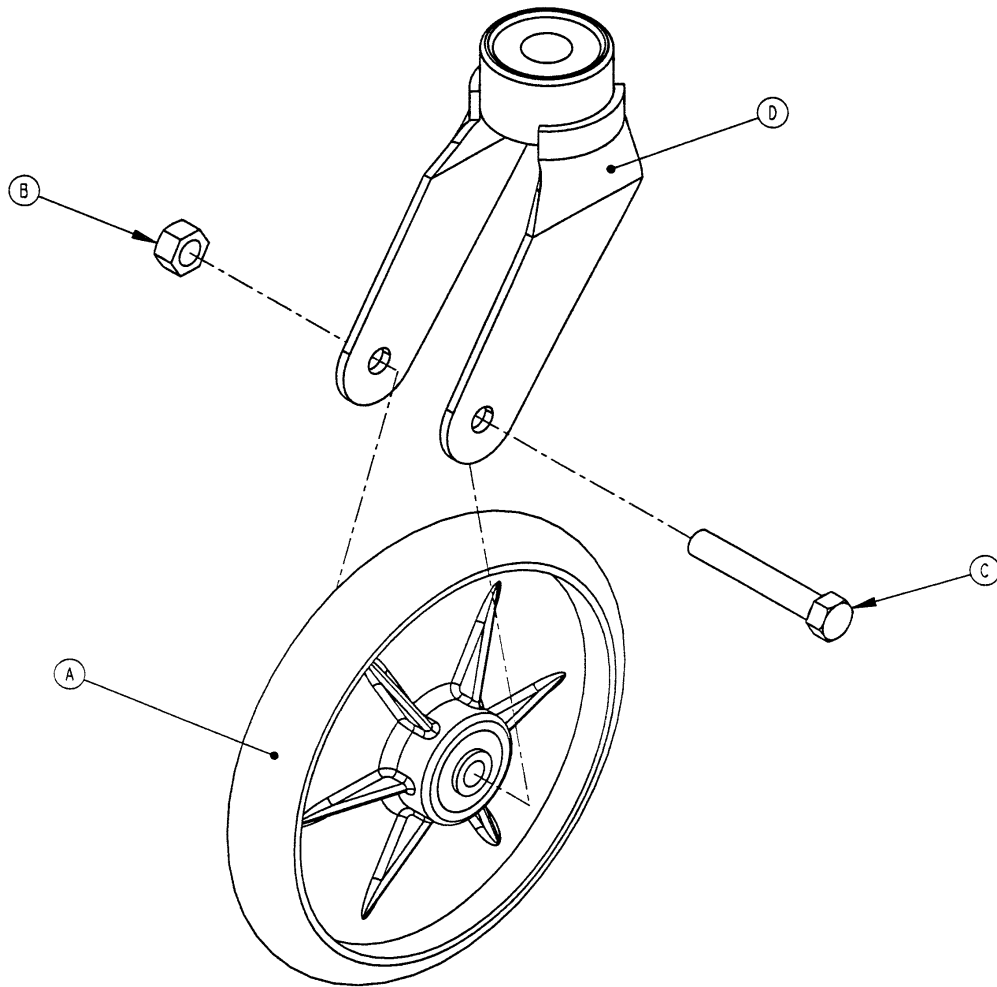


A	0715-002-025	1
B	0016-060-000	1
C	0003-099-000	1
D	3001-200-076	1


Rev A

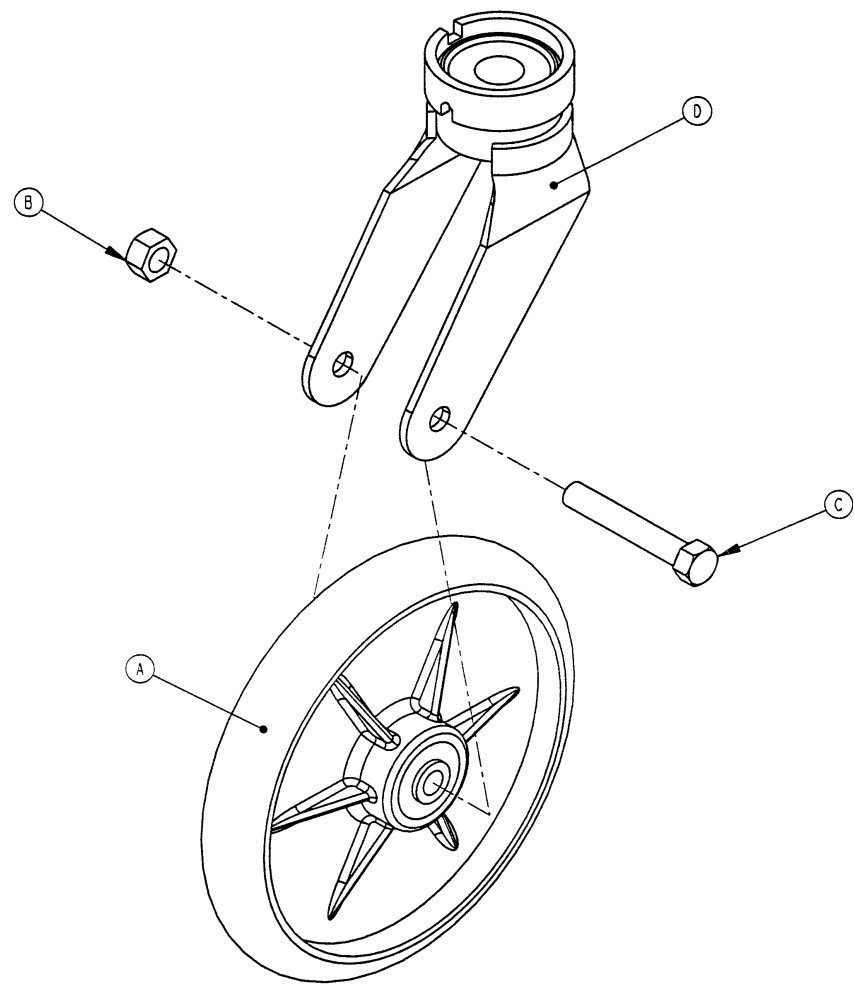



A	0715-002-025	1
B	0016-060-000	1
C	0003-099-000	1
D	3001-200-081	1

Rev A

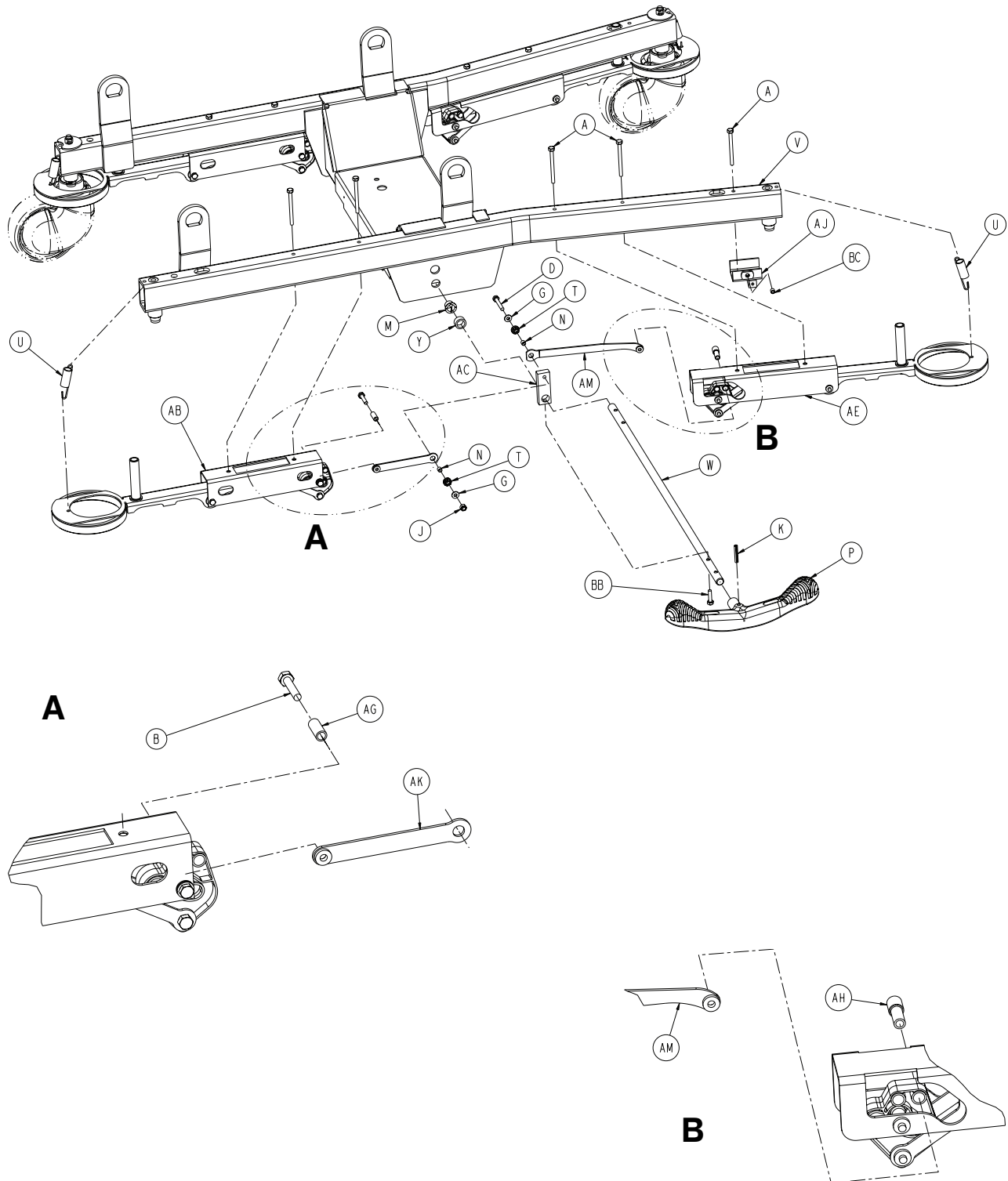


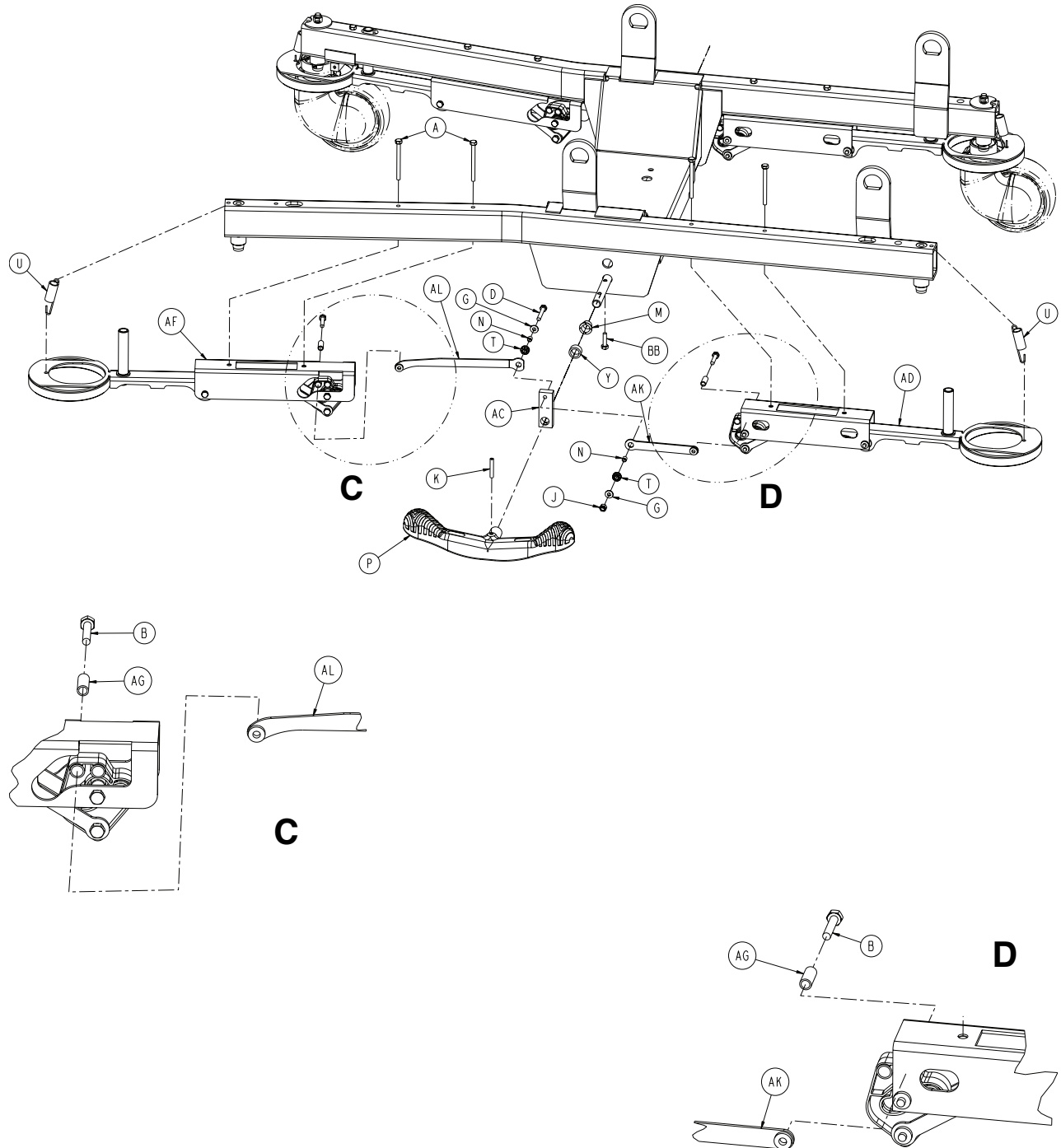

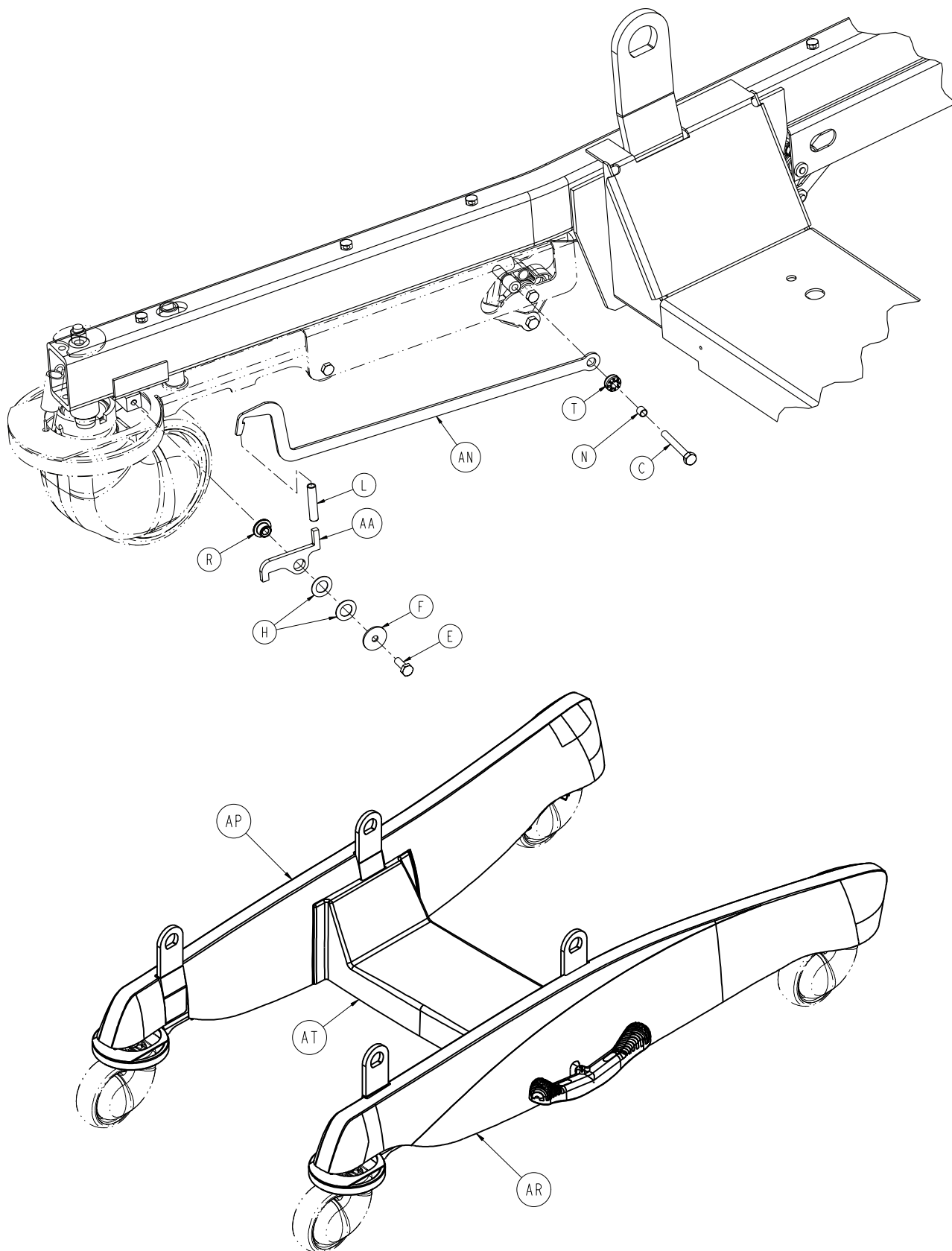

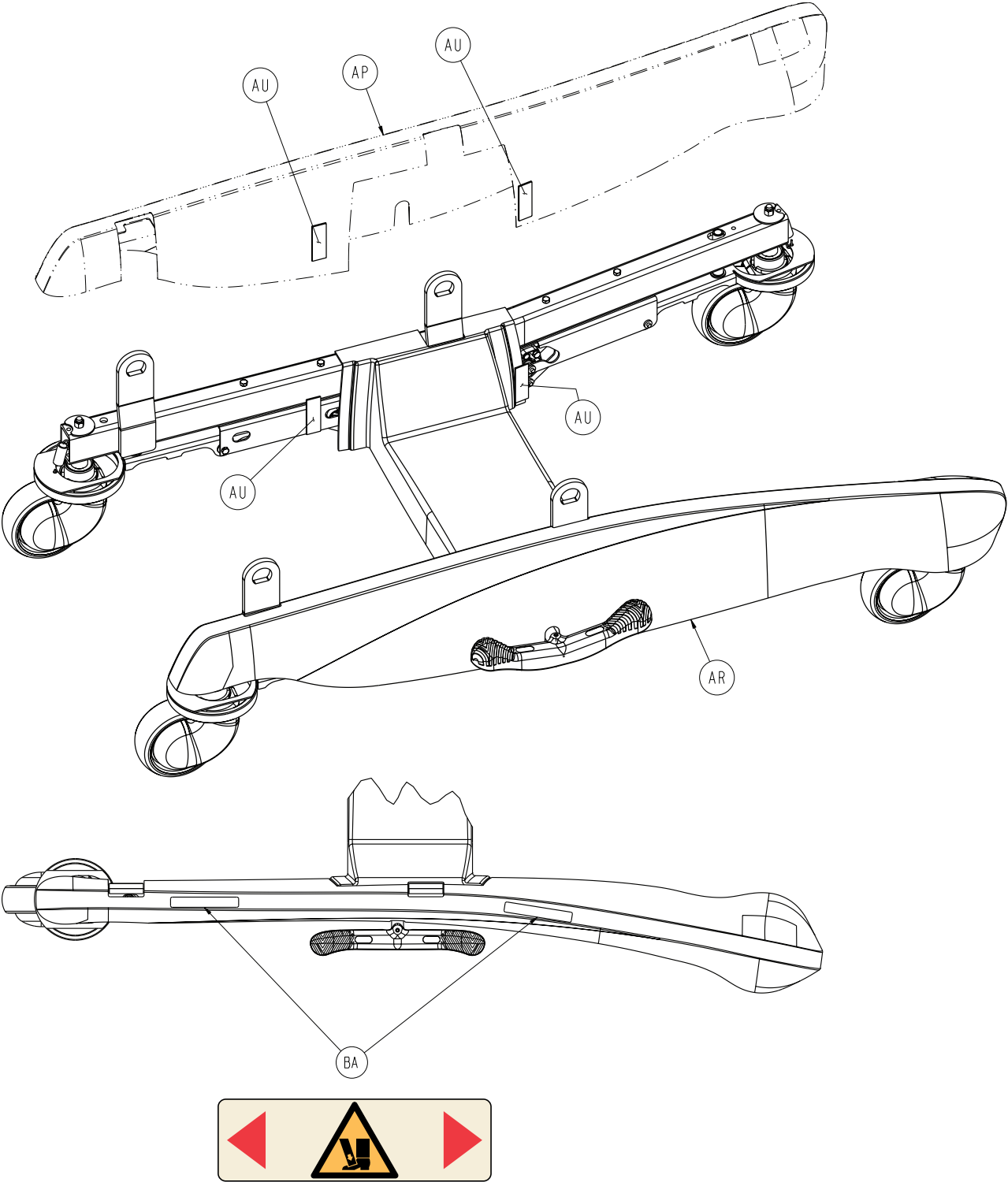

stryker

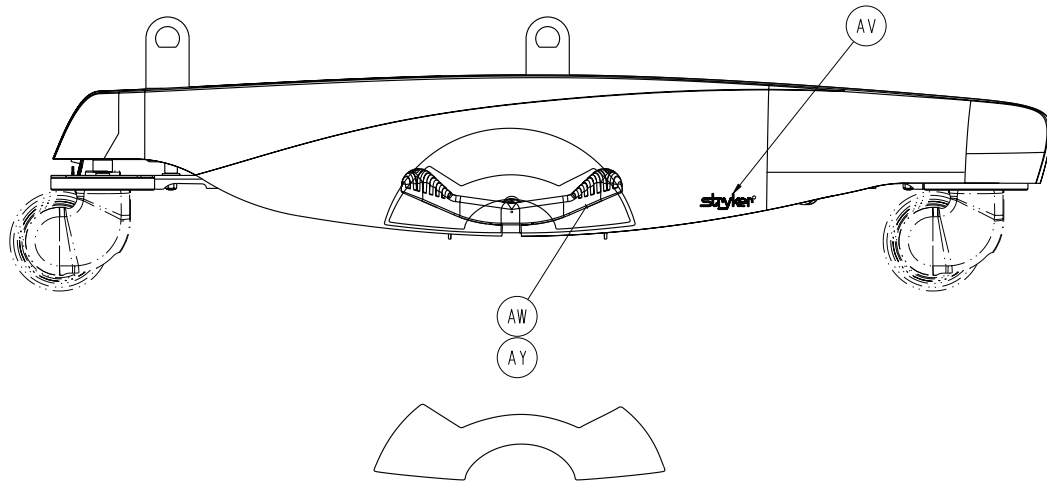


4701-201-000

A	0003-056-000	9
B	0003-139-000	3
C	0003-076-000	1
D	0003-078-000	2
E	0003-214-000	3
F	0011-053-000	1
G	0011-063-000	2
H	0014-007-000	2
J	0016-028-000	2
K	0026-067-000	2
L	0038-524-000	1
M	0052-509-000	2
N	0715-001-333	5
P	1033-001-233	2
R	3000-300-099	1
T	3000-300-442	5
U	3002-200-310	4
V	4701-101-010	1
W	4701-001-016	1
Y	4701-001-018	2
AA	4701-001-019	1
AB	4701-101-020	1
AC	4701-001-021	2
AD	4701-101-025	1
AE	4701-001-030	1
AF	4701-001-035	1
AG	4701-001-053	3
AH	4701-001-057	1
AJ	4701-001-065	1
AK	4701-101-066	2
AL	4701-001-067	1
AM	4701-001-069	1
AN	4701-001-074	1
AP	4701-101-085	1
AR	4701-101-090	1
AT	4701-001-091	1
AU	0029-009-000	4
AV	1040-010-017	2
AW	4701-090-051	1

4701-201-000


AY	4701-090-052	1
BA	4701-109-953	4
BB	0003-382-000	2
BC	0021-140-000	1

Rev B

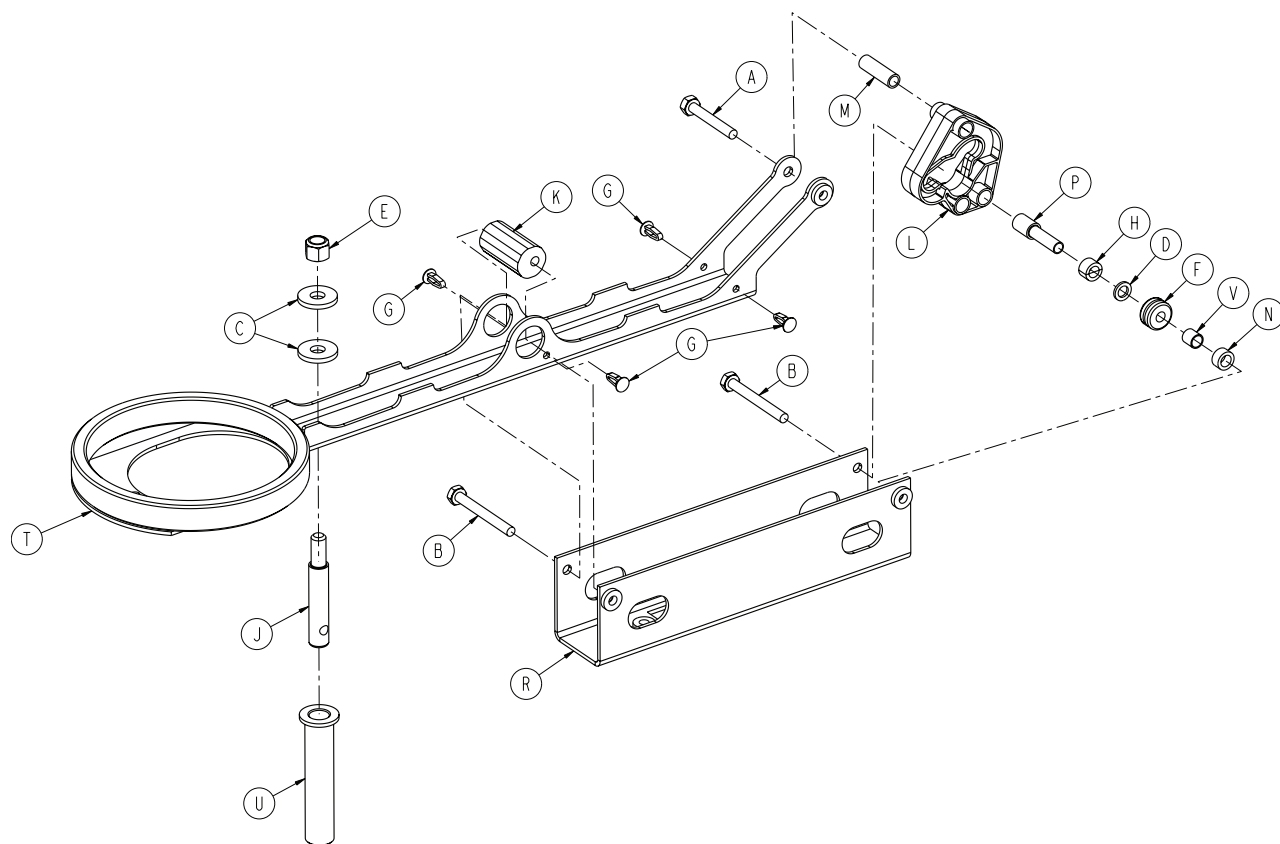


A	0003-078-000	1
B	0003-079-000	2
C	0011-262-000	2
D	0014-003-000	1
E	0016-035-000	1
F	0030-053-000	1
G	0049-005-000	4
H	0081-409-000	1
J	3000-200-318	1
K	4701-001-023	1
L	4701-001-042	1
M	4701-001-052	1
N	4701-001-056	1
P	4701-001-058	1
R	4701-001-078	1
T	4701-001-080	1
U	4701-001-083	1
V	0081-259-000	1

Rev A

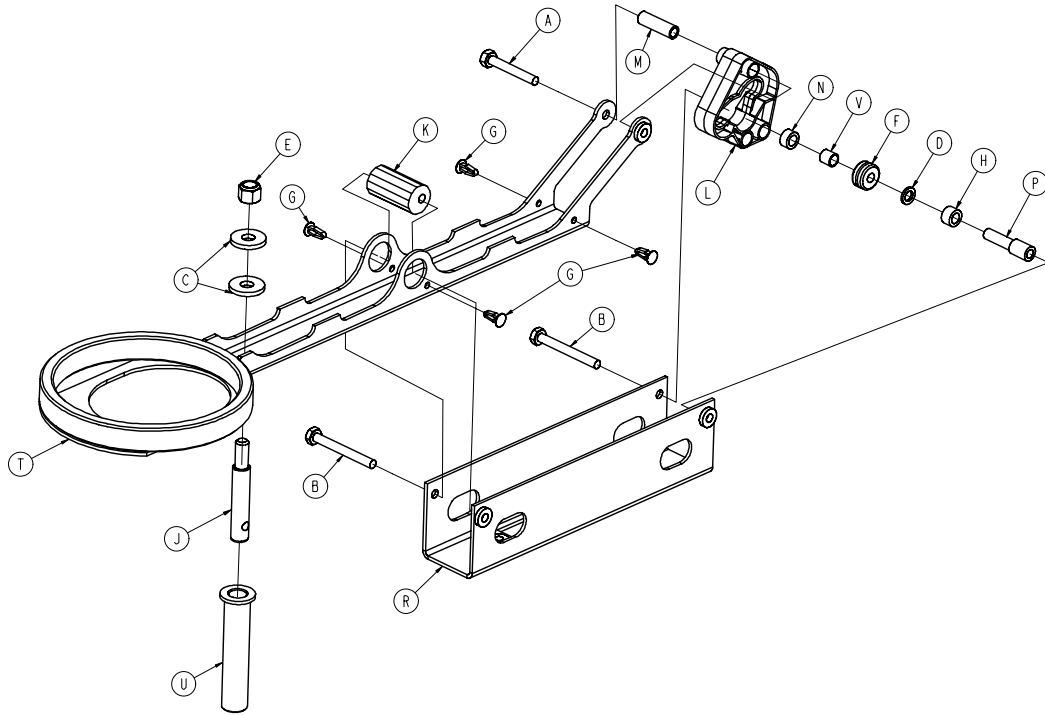


A	0003-078-000	1
B	0003-079-000	2
C	0011-262-000	2
D	0014-003-000	1
E	0016-035-000	1
F	0030-053-000	1
G	0049-005-000	4
H	0081-409-000	1
J	3000-200-318	1
K	4701-001-023	1
L	4701-001-042	1
M	4701-001-052	1
N	4701-001-056	1
P	4701-001-058	1
R	4701-001-078	1
T	4701-001-080	1
U	4701-001-083	1
V	0081-259-000	1

Rev B

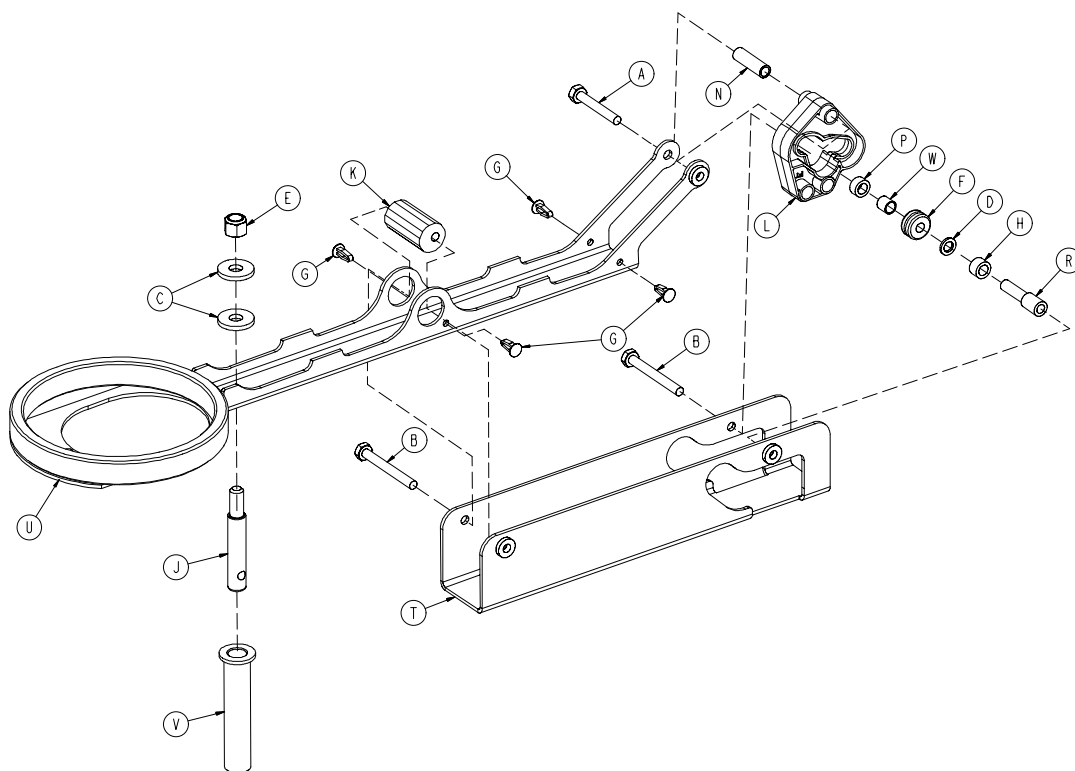


A	0003-078-000	1
B	0003-079-000	2
C	0011-262-000	2
D	0014-003-000	1
E	0016-035-000	1
F	0030-053-000	1
G	0049-005-000	4
H	0081-409-000	1
J	3000-200-318	1
K	4701-001-023	1
L	4701-001-042	1
N	4701-001-052	1
P	4701-001-056	1
R	4701-001-058	1
T	4701-001-072	1
U	4701-001-080	1
V	4701-001-083	1
W	0081-259-000	1


Rev B

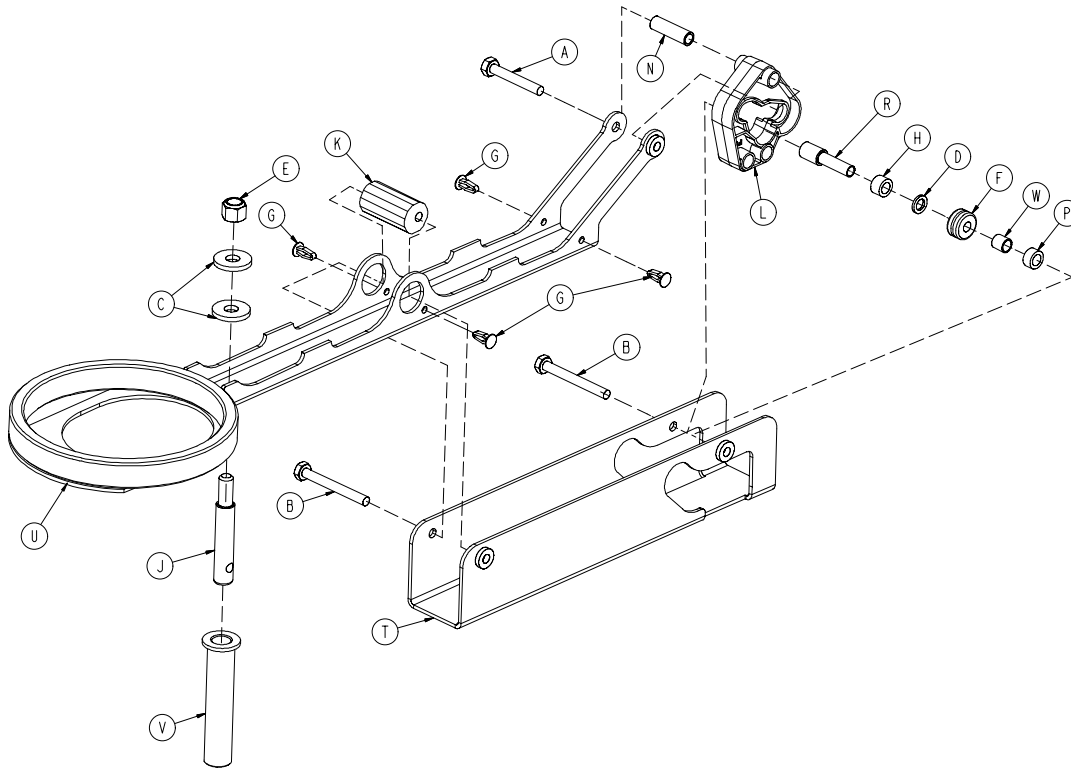



A	0003-078-000	1
B	0003-079-000	2
C	0011-262-000	2
D	0014-003-000	1
E	0016-035-000	1
F	0030-053-000	1
G	0049-005-000	4
H	0081-409-000	1
J	3000-200-318	1
K	4701-001-023	1
L	4701-001-042	1
N	4701-001-052	1
P	4701-001-056	1
R	4701-001-058	1
T	4701-001-072	1
U	4701-001-080	1
V	4701-001-083	1
W	0081-259-000	1

Rev C

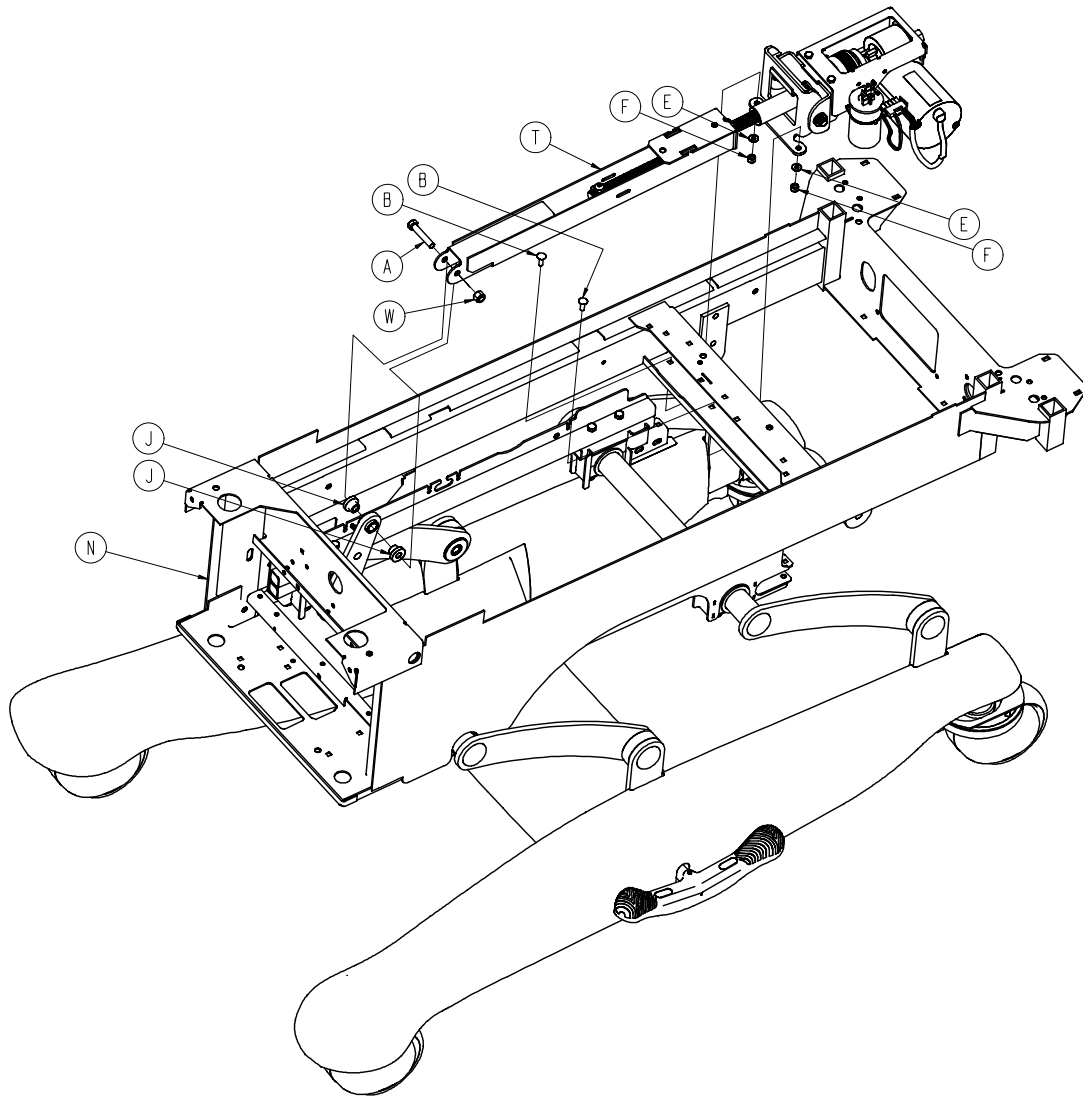

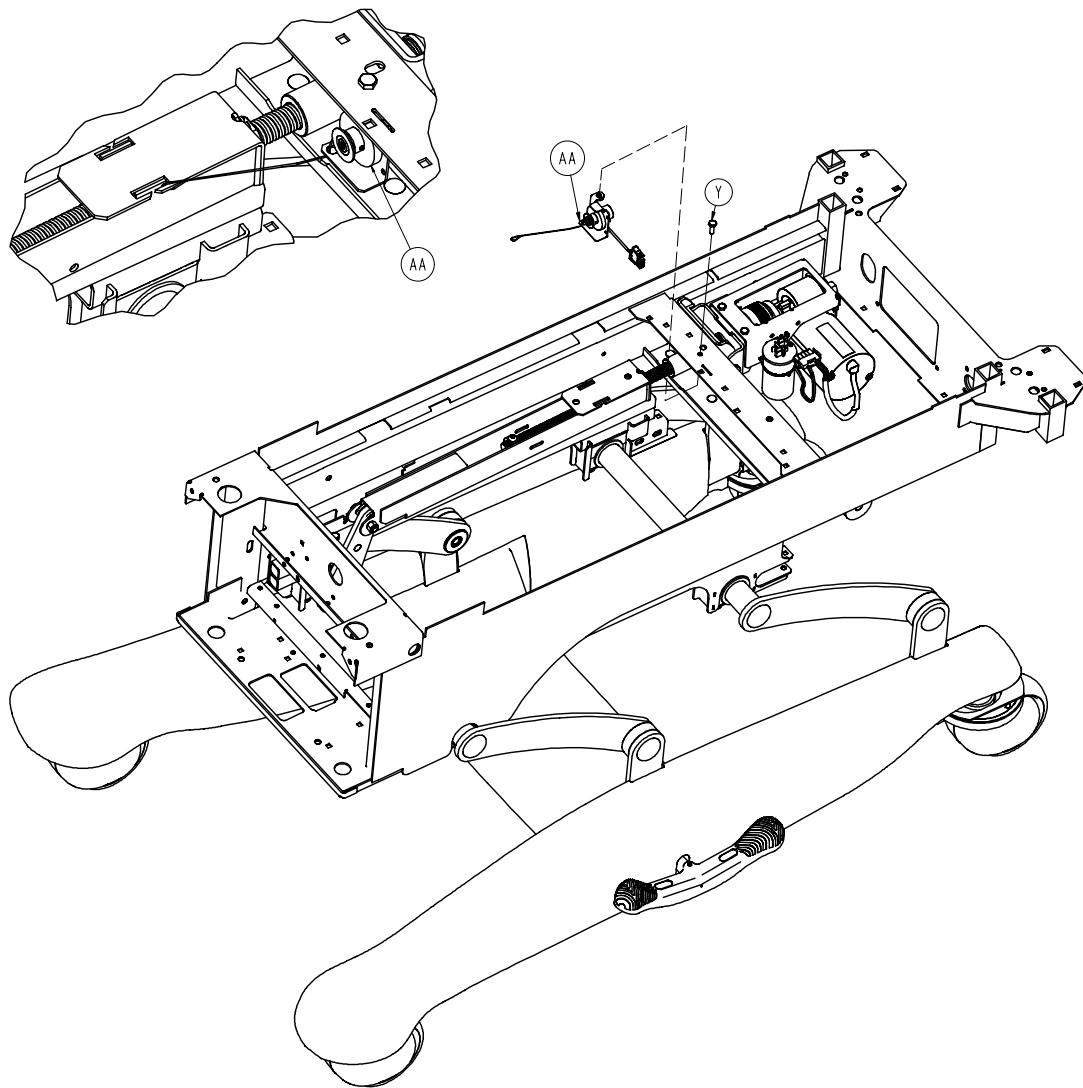

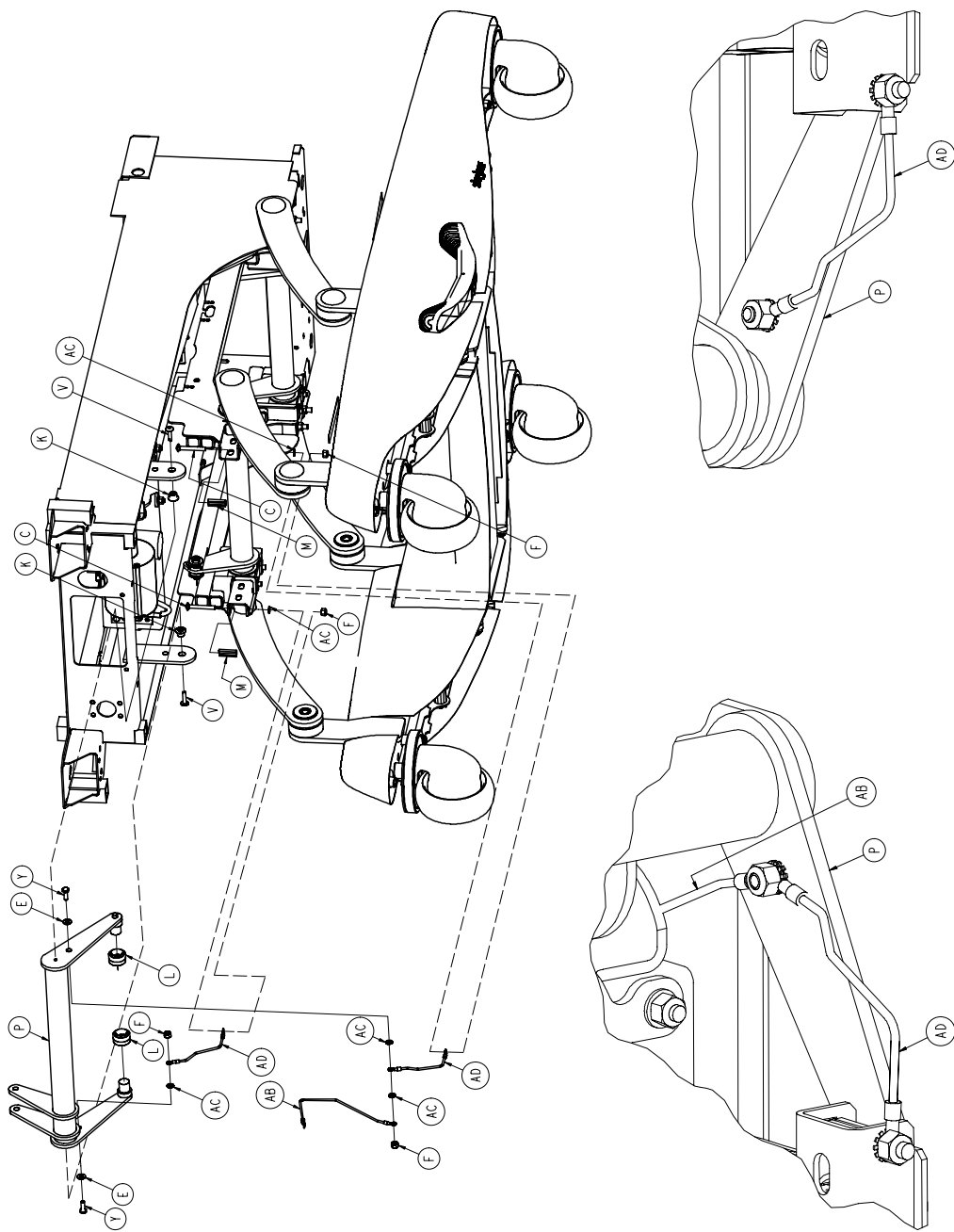


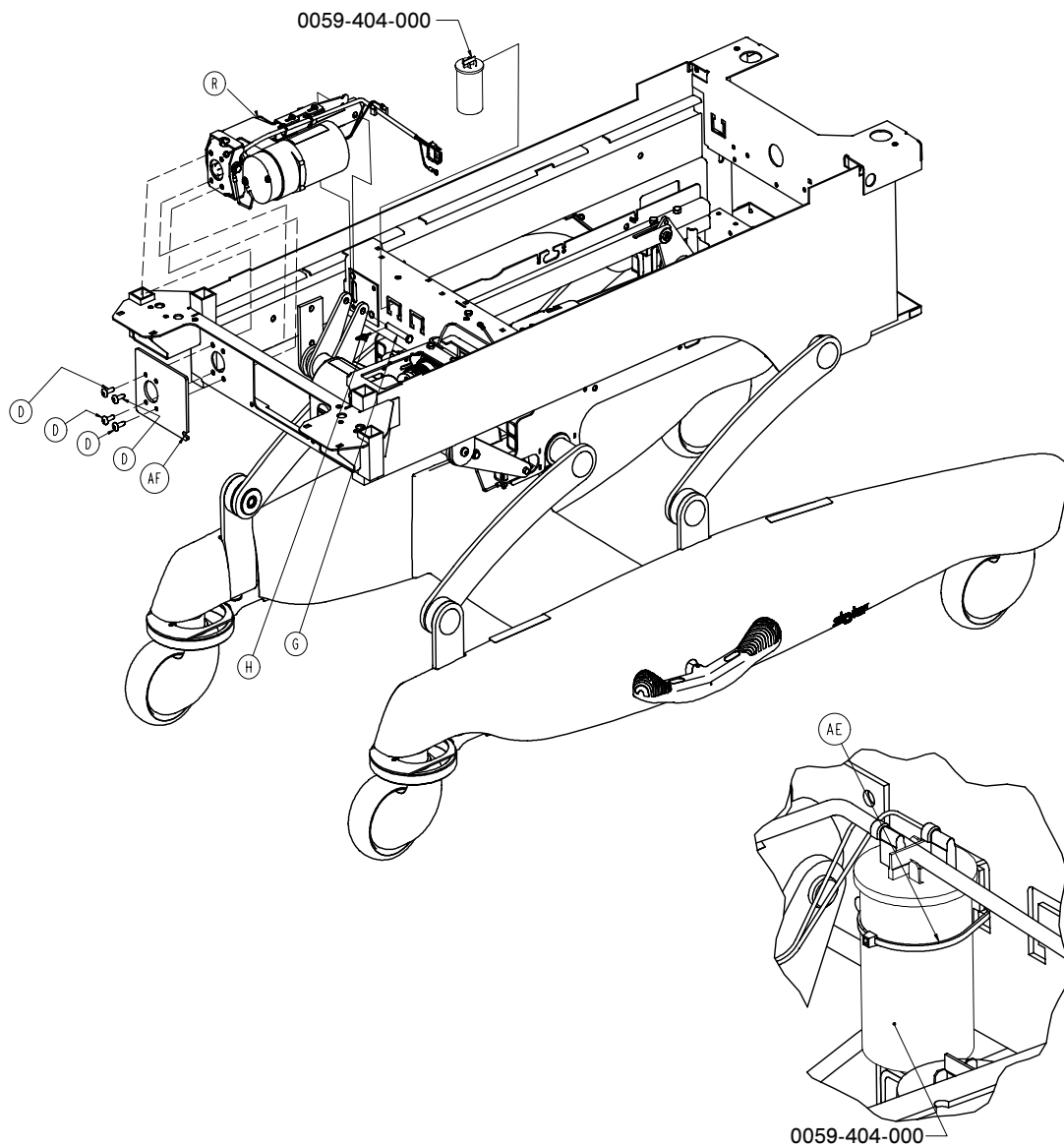



4712-032-100


A	0003-347-000	1
B	0005-017-000	2
C	0005-024-000	2
D	0007-065-000	4
E	0011-063-000	4
F	0016-028-000	6
G	0026-353-000	1
H	0027-022-000	1
J	0081-408-000	2
K	3000-300-001	2
L	3000-300-353	2
M	3001-300-004	2
N	4701-032-000	1
P	4701-032-010	1
R	4712-032-015	1
T	4712-032-055	1
V	0007-085-000	2
W	0016-035-000	1
Y	0003-214-000	3
AA	4701-080-025	1
AB	4701-080-018	1
AC	0013-010-000	5
AD	3002-300-870	2
AE	0059-086-000	2
AF	4701-032-087	1

Rev G

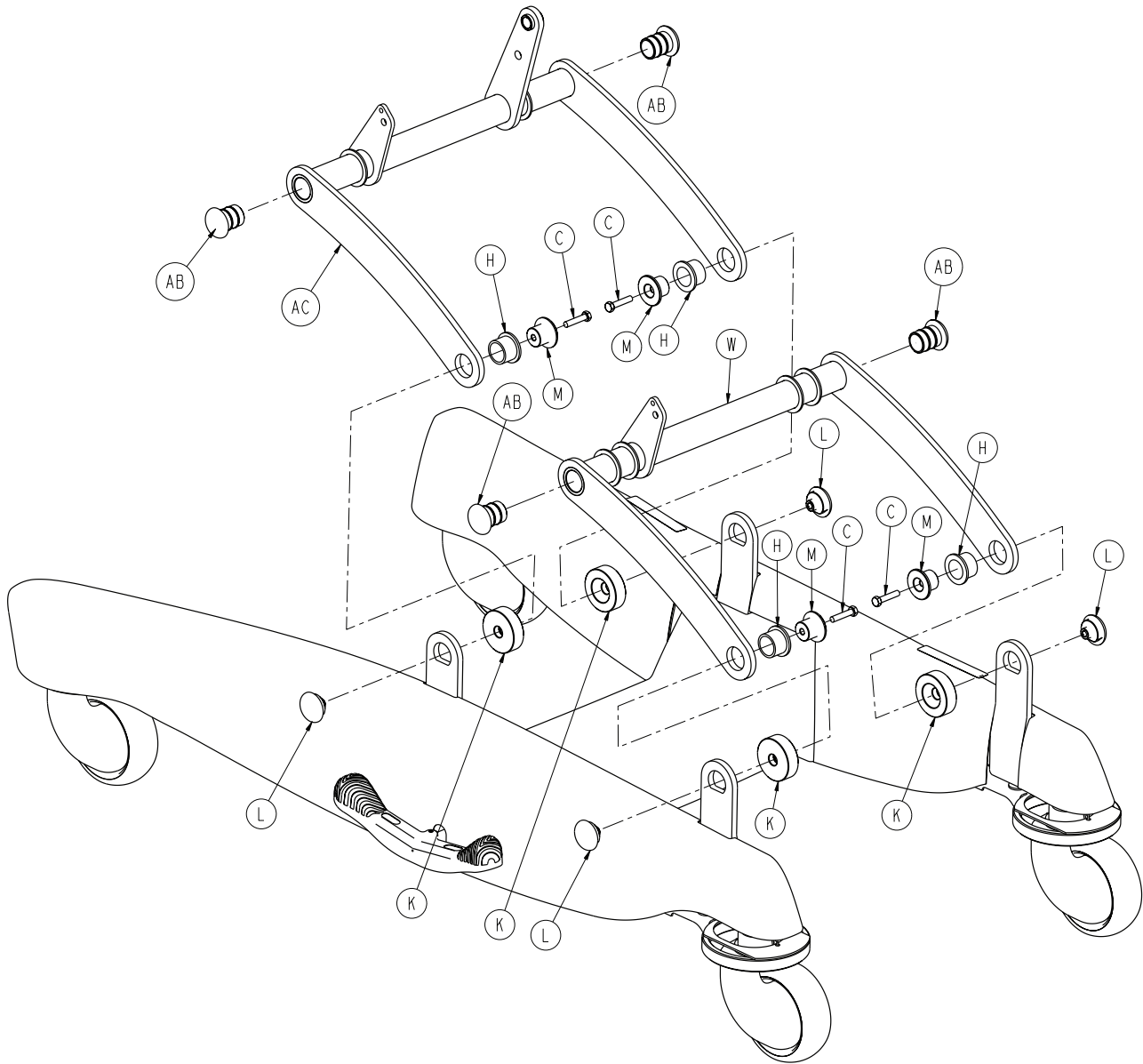

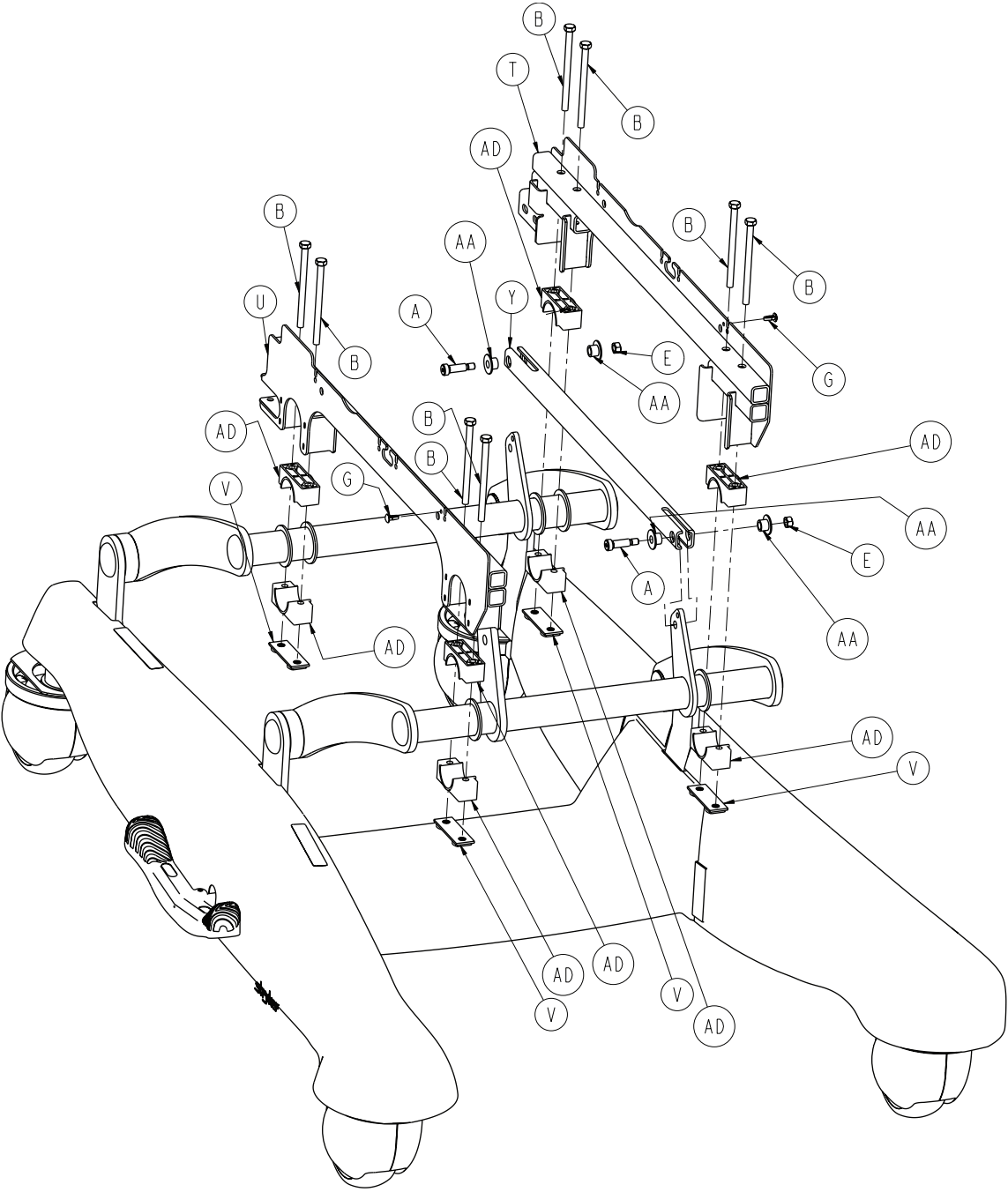

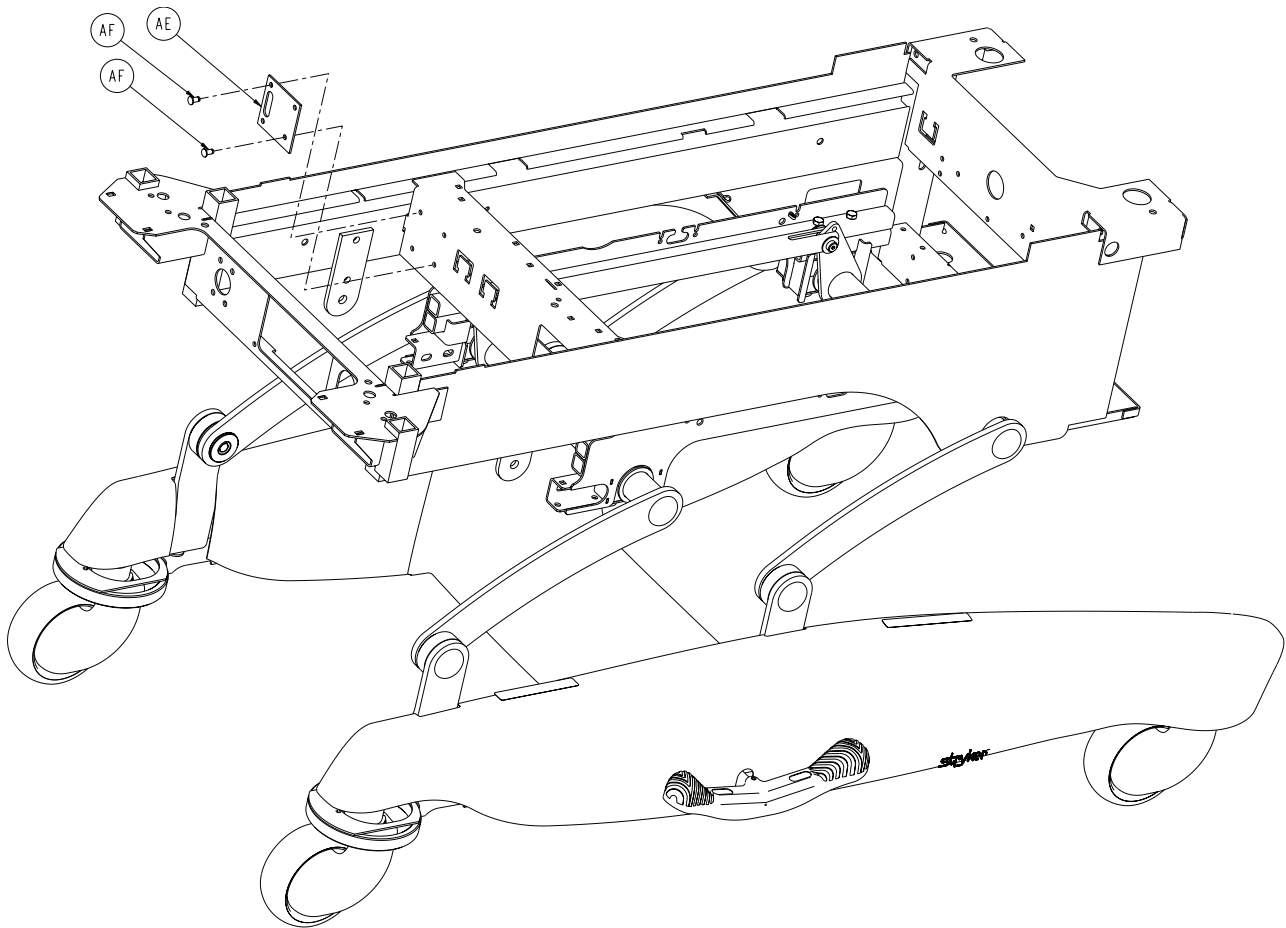



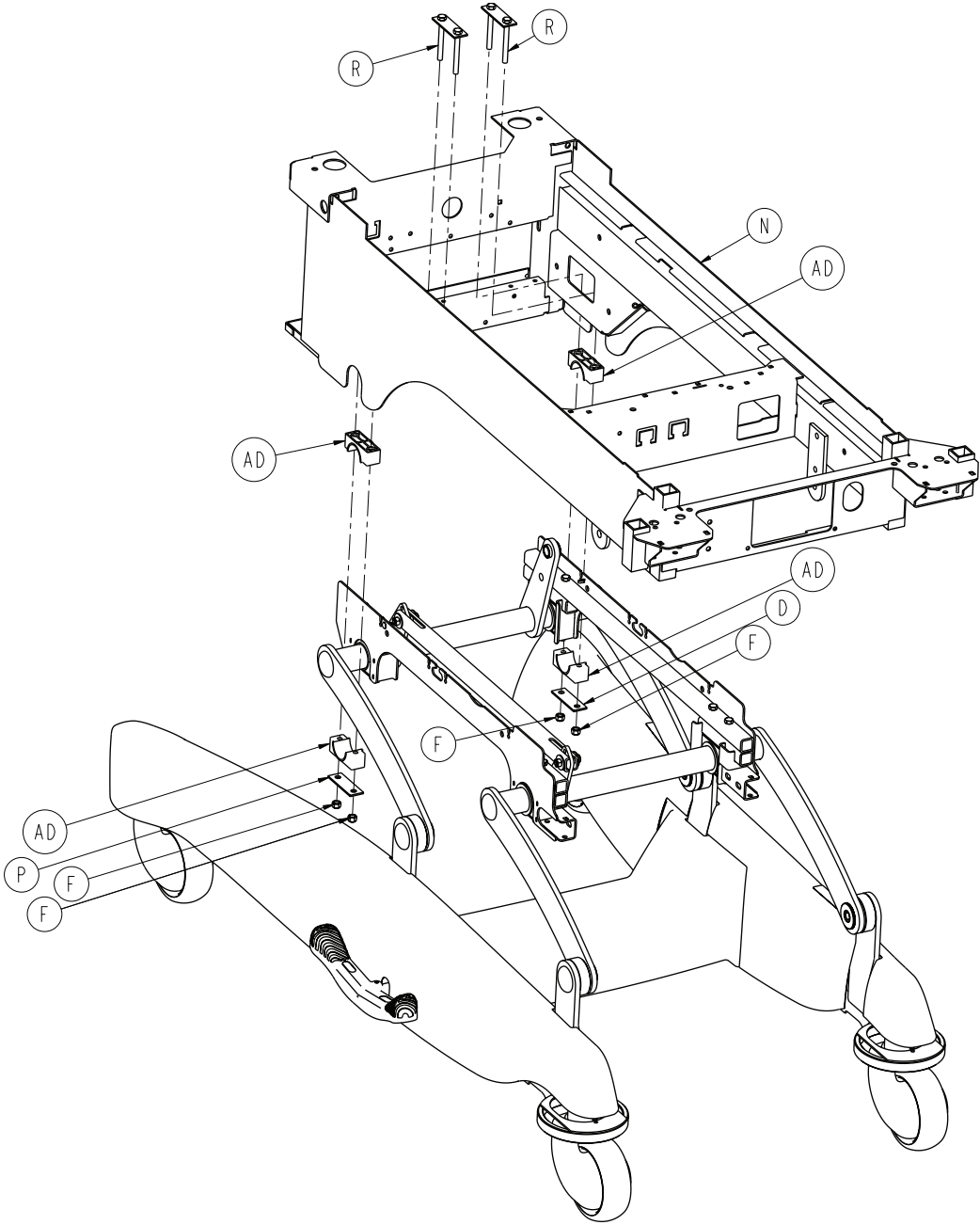


4701-032-000

A	0008-024-000	2
B	0003-371-000	8
C	0003-374-000	4
E	0016-020-000	2
F	0016-036-000	4
G	0052-904-000	2
H	0081-406-000	4
K	4701-001-047	4
L	4701-001-048	4
M	4701-001-049	4
N	4701-030-010	1
P	4701-030-179	2
R	4701-030-180	2
T	4701-232-008	1
U	4701-232-009	1
V	4701-032-026	4
W	4701-032-030	1
Y	4701-232-031	1
AA	4701-232-032	4
AB	4701-032-039	4
AC	4701-032-040	1
AD	5000-030-237	12
AE	4701-030-087	1
AF	0025-120-000	2


Rev H

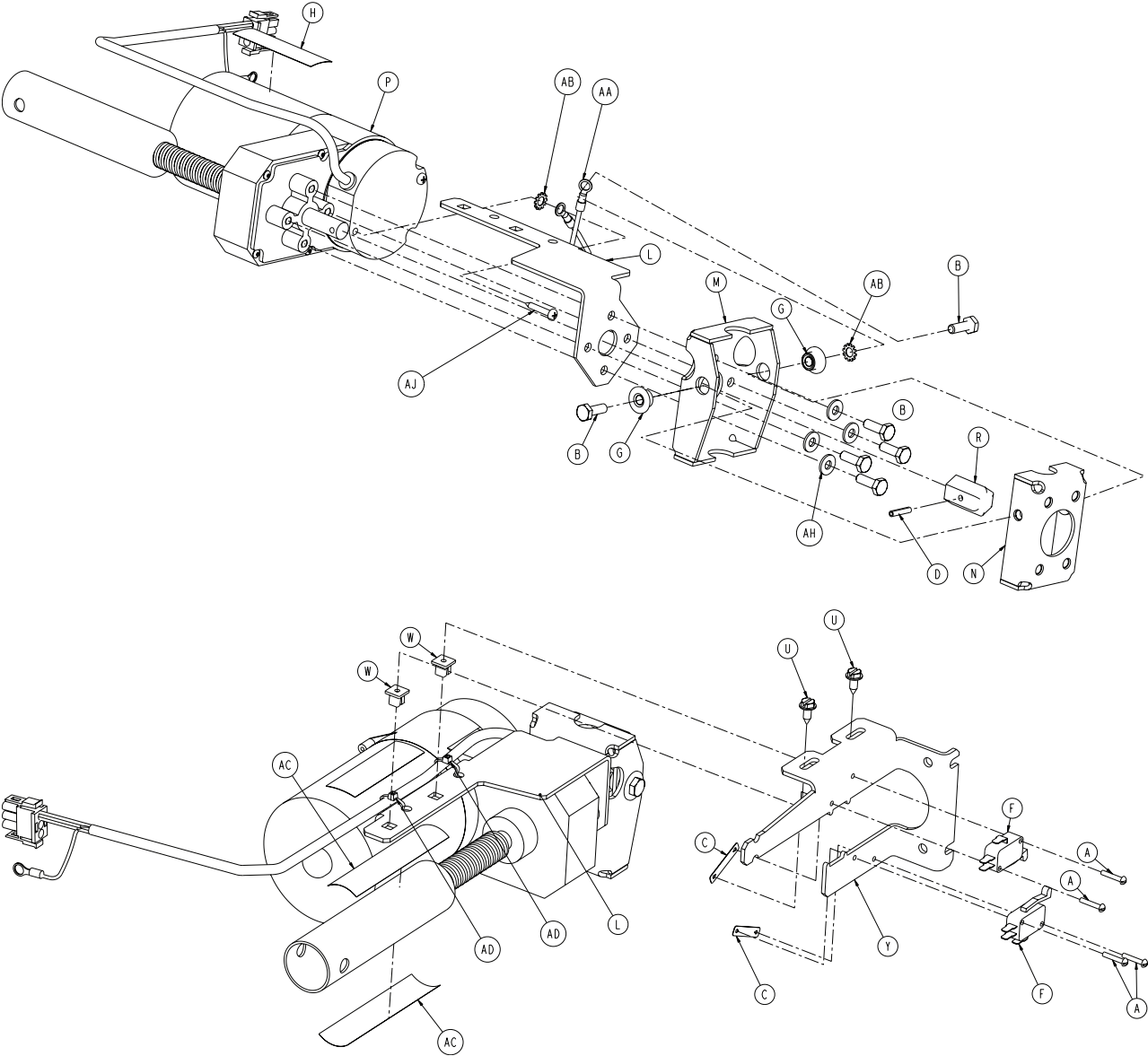

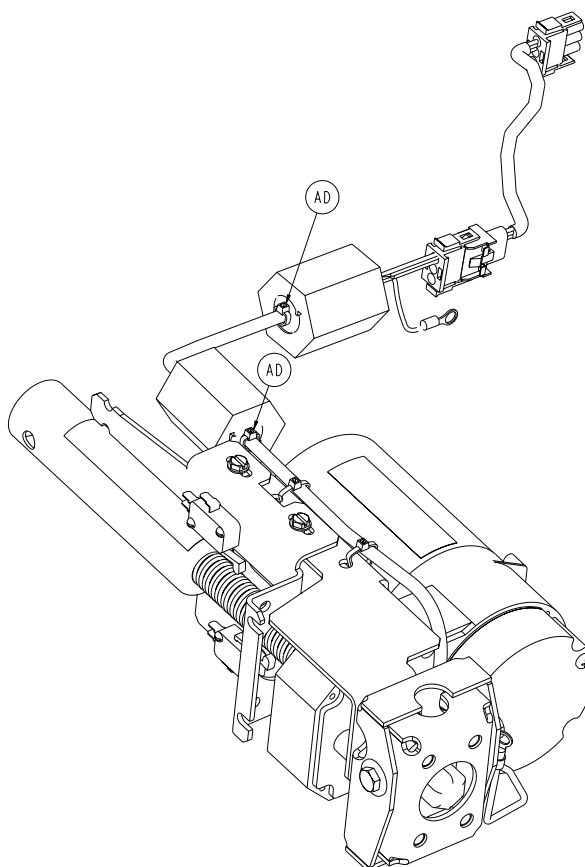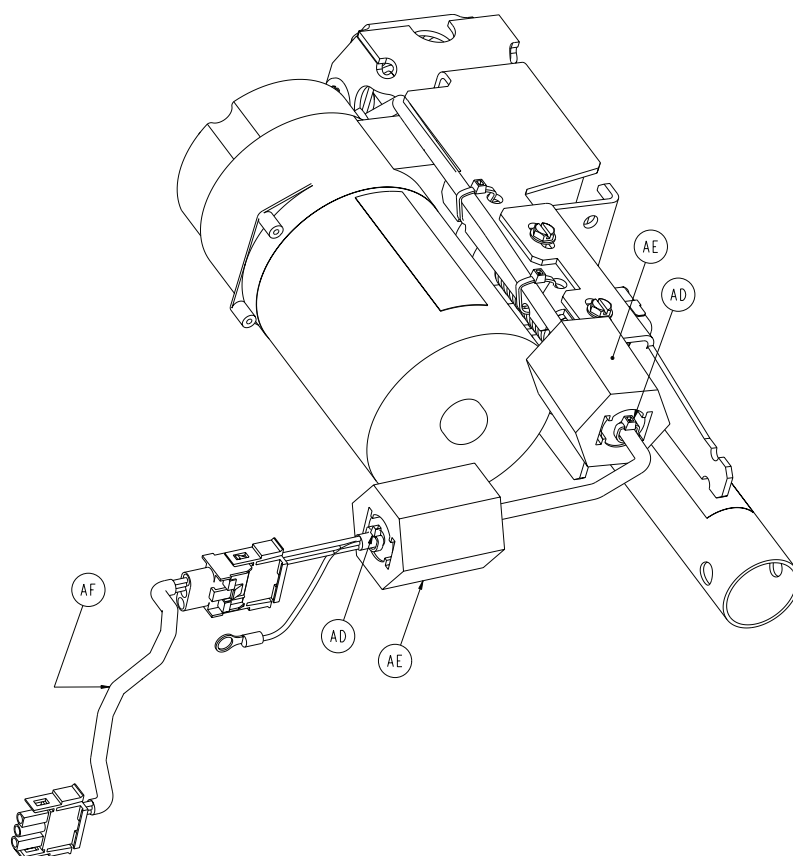


A	0002-051-000	4
B	0003-214-000	6
C	0016-069-000	2
D	0026-085-000	1
F	3000-300-041	2
G	3000-300-099	2
H	4701-090-056	1
L	4701-232-016	1
M	4701-232-017	1
N	4701-032-018	1
P	4712-116-002	1
R	4701-032-049	1
U	0023-025-000	2
W	3000-300-002	2
Y	4701-032-019	1
AA	3002-300-870	1
AB	0013-010-000	2
AC	0044-056-000	8"
AD	0059-086-000	6
AE	0059-192-000	2
AF	4701-080-029	1
AH	0011-063-000	4
AJ	0002-111-000	1


Rev D

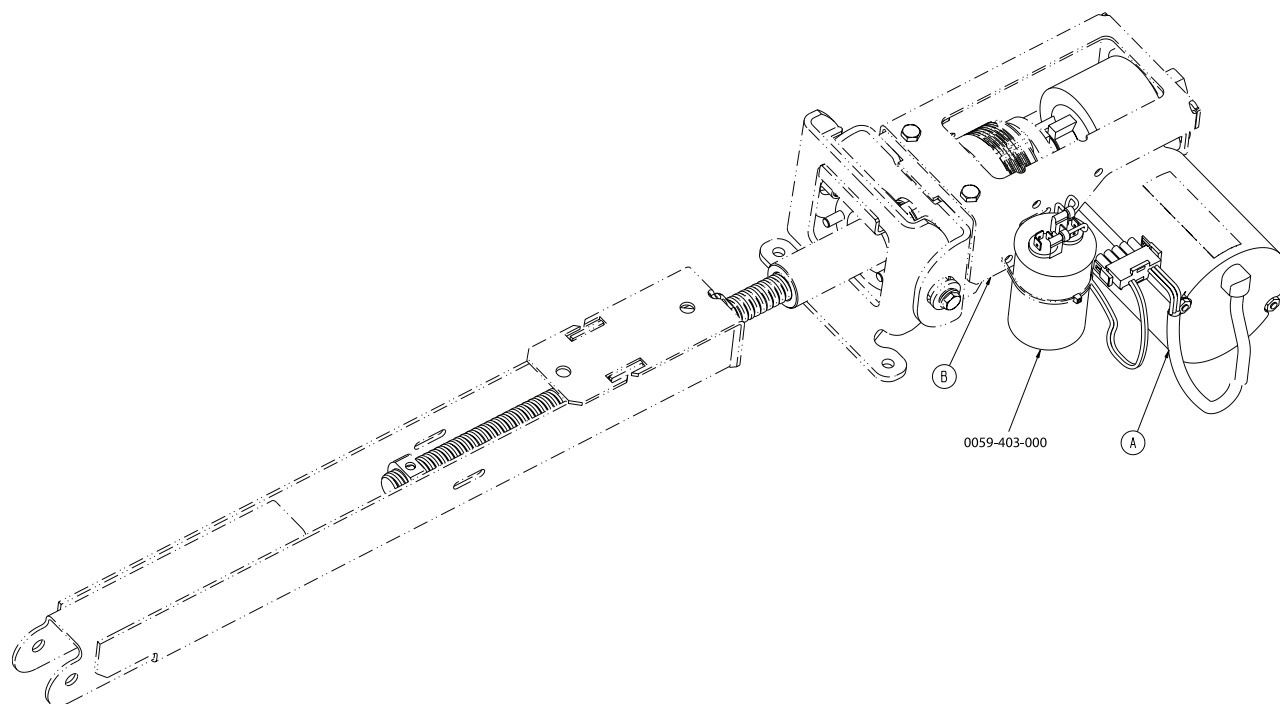



A	4712-116-003	1
B	4701-032-075	1

Rev N

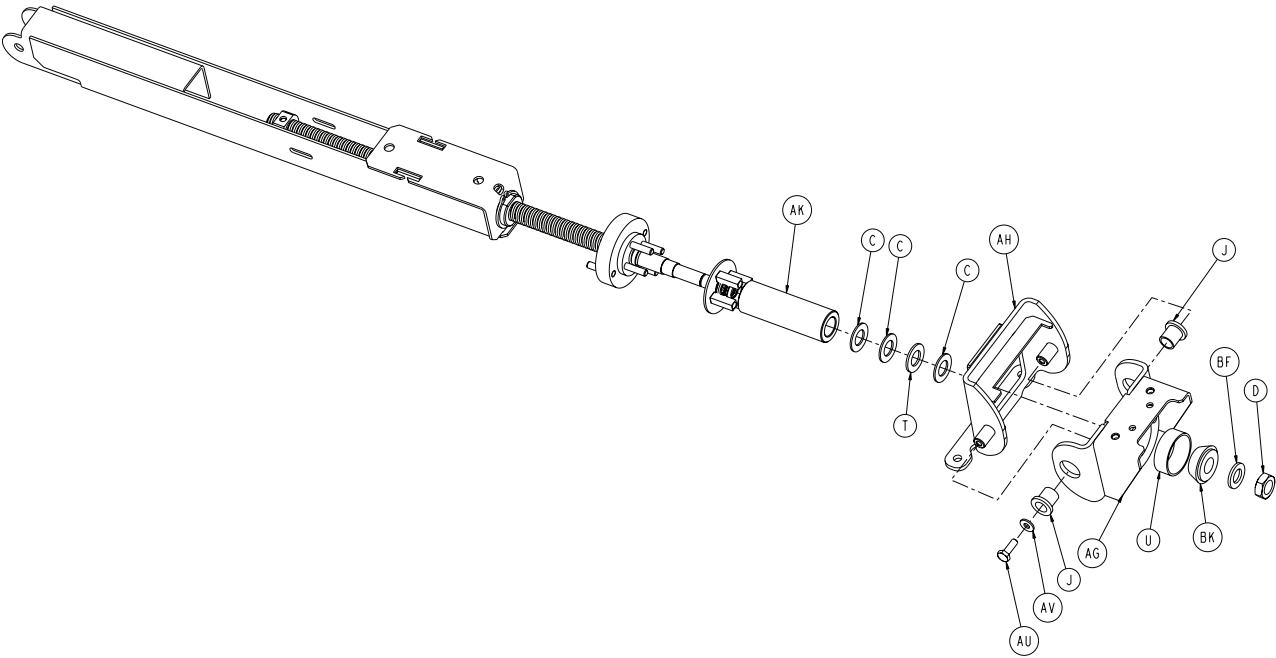


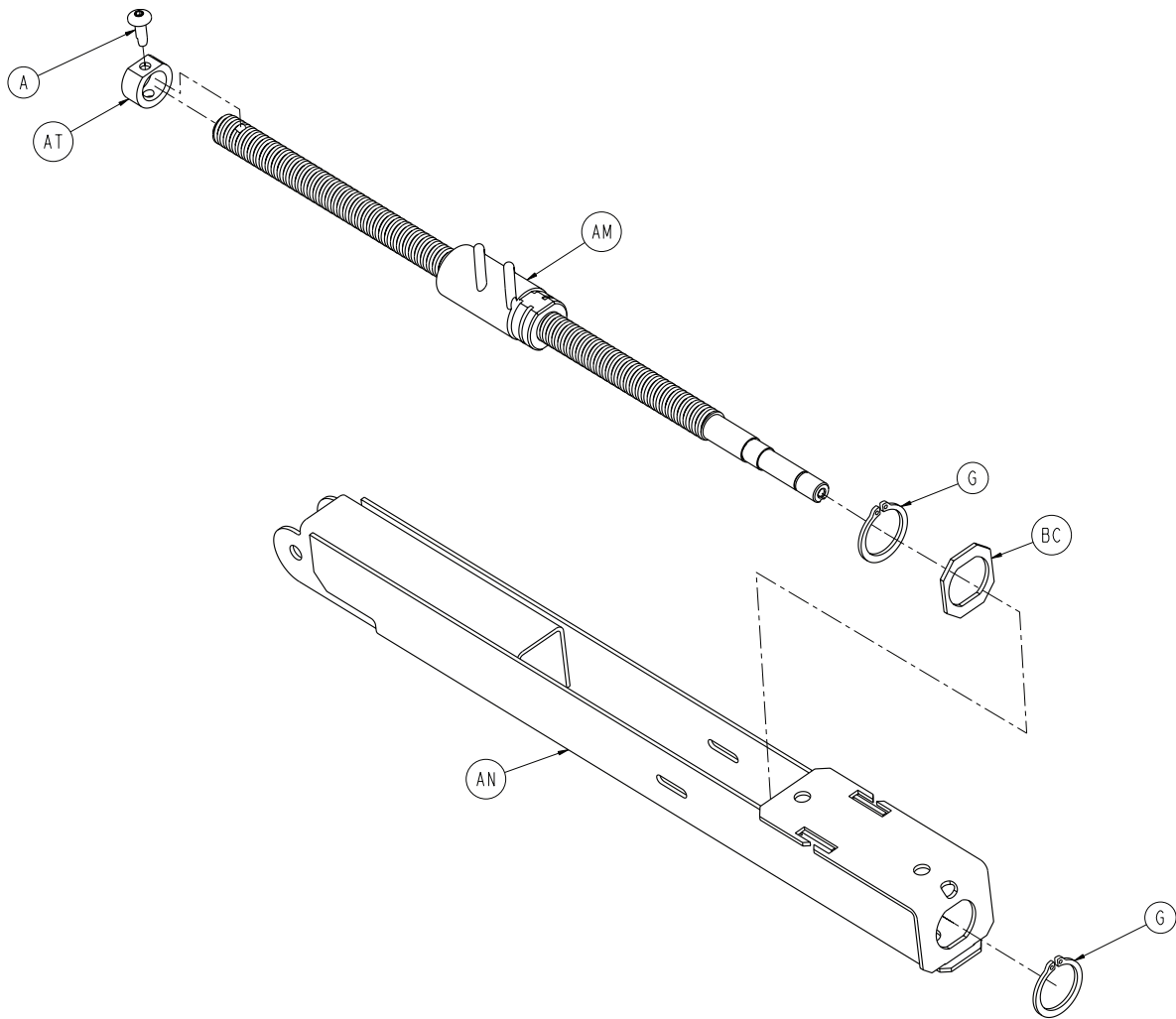

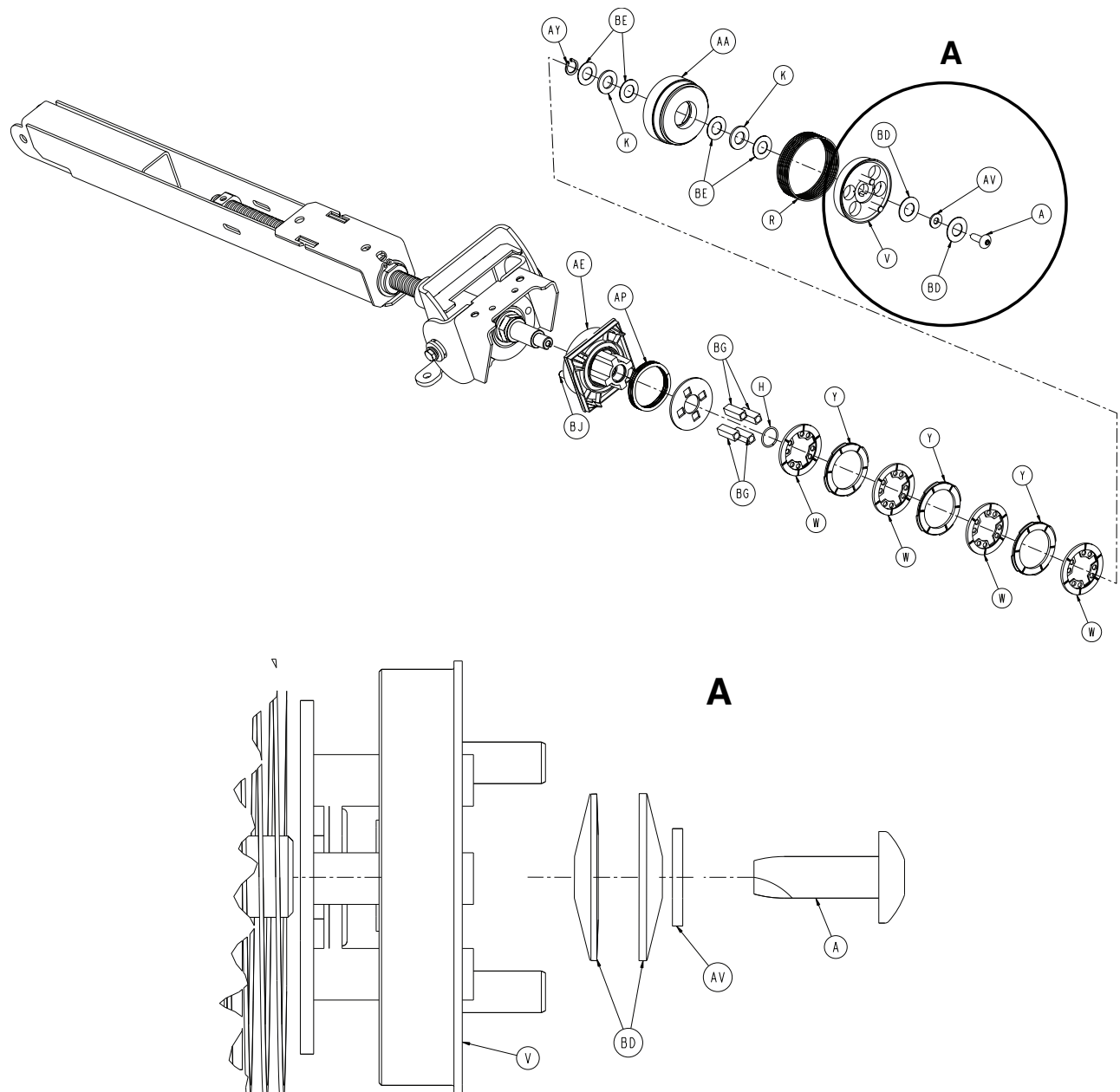

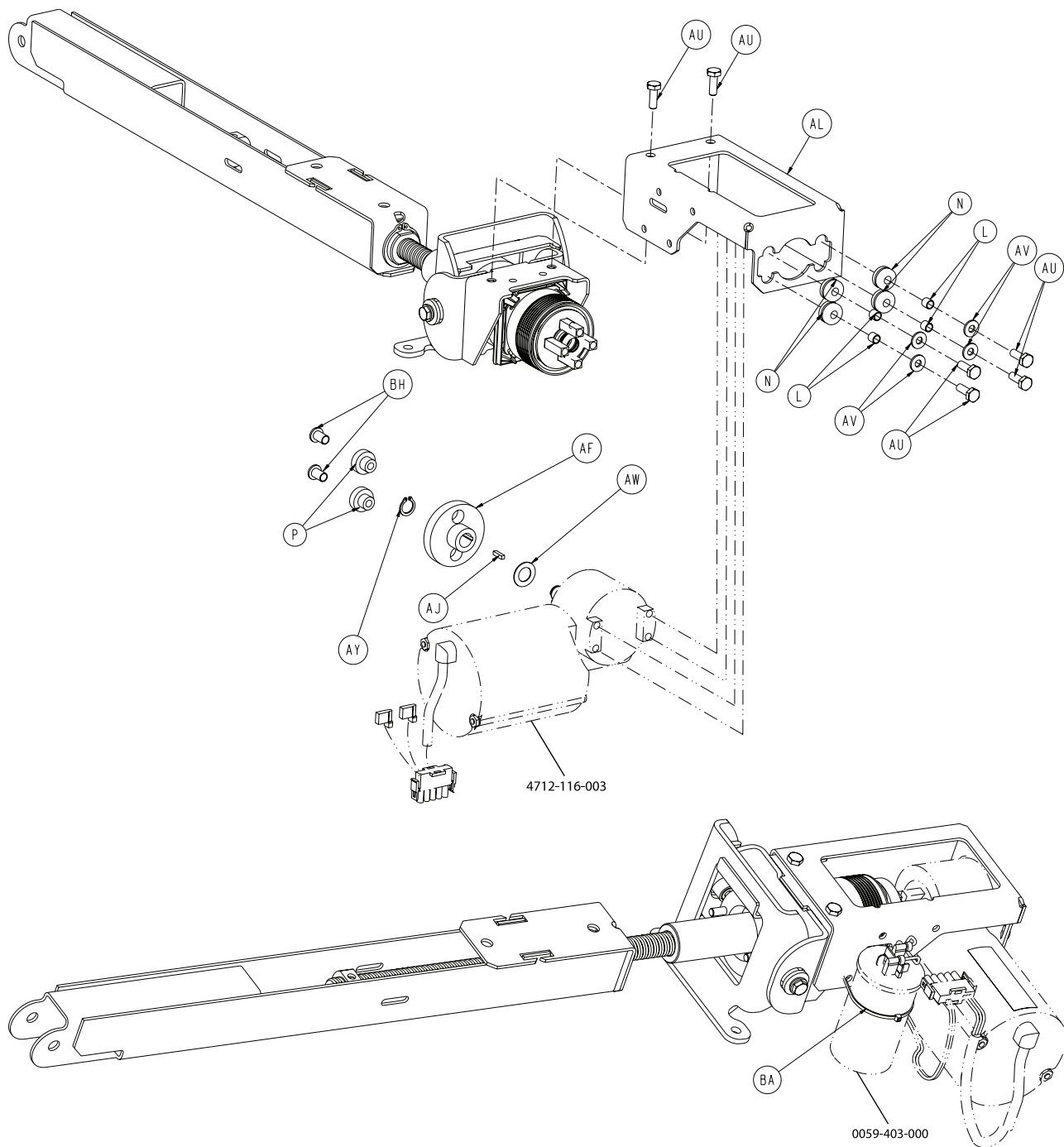

4701-032-075

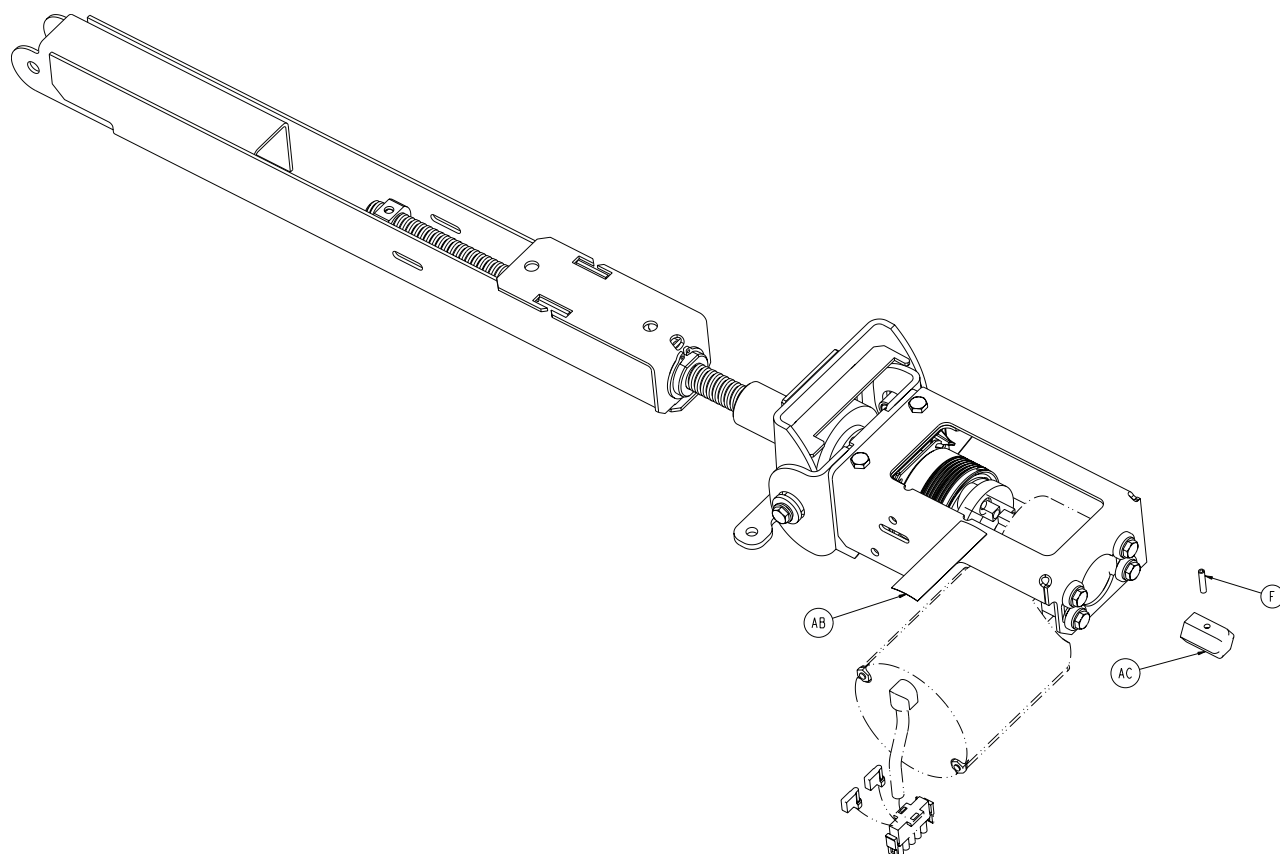


A	0007-065-000	2
C	0014-083-000	3
D	0015-075-000	1
F	0026-085-000	1
G	0028-120-000	2
H	0028-036-000	1
J	0081-008-000	2
K	0081-212-000	1
L	0715-001-333	4
N	0030-053-000	4
P	3000-300-455	2
R	3000-300-465	1
T	3000-300-466	1
U	3000-300-471	1
V	4701-032-090	1
W	3001-300-551	4
Y	3001-300-552	3
AA	4701-032-076	1
AB	4701-090-056	1
AC	4701-032-049	1

4701-032-075


AD	4701-232-091	1
AE	4701-032-093	1
AF	4701-032-058	1
AG	4701-032-060	1
AH	4701-032-067	1
AJ	4701-032-071	1
AK	4701-032-072	1
AL	4701-032-073	1
AM	4701-032-078	1
AN	4701-032-080	1
AP	3001-300-563	1
AT	5000-030-255	1
AU	0003-214-000	7
AV	0011-063-000	6
AW	0014-007-000	1
AY	0028-131-000	2
BA	0059-086-000	1
BC	4701-032-086	1
BD	0014-123-000	2
BE	0011-600-001	4
BF	4701-032-088	1
BG	4701-032-096	4
BH	0052-229-000	2
BJ	0044-056-000	2
BK	3000-300-444	1


Rev C

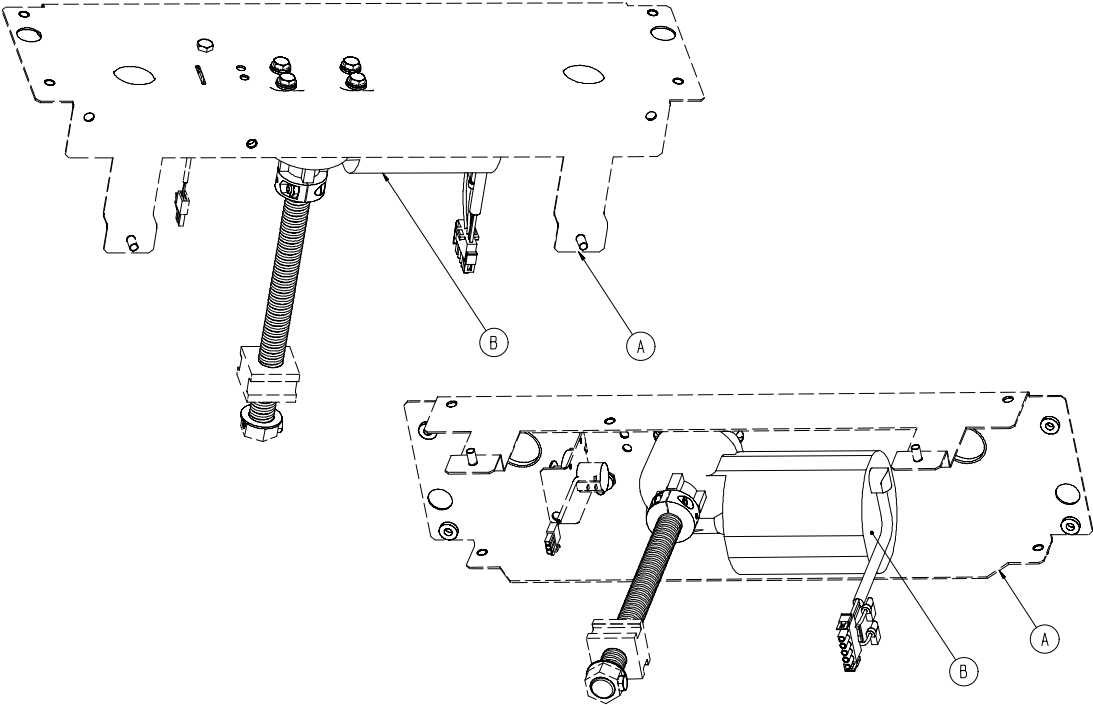

A	4701-040-000	1
B	4712-116-001	1


Rev C

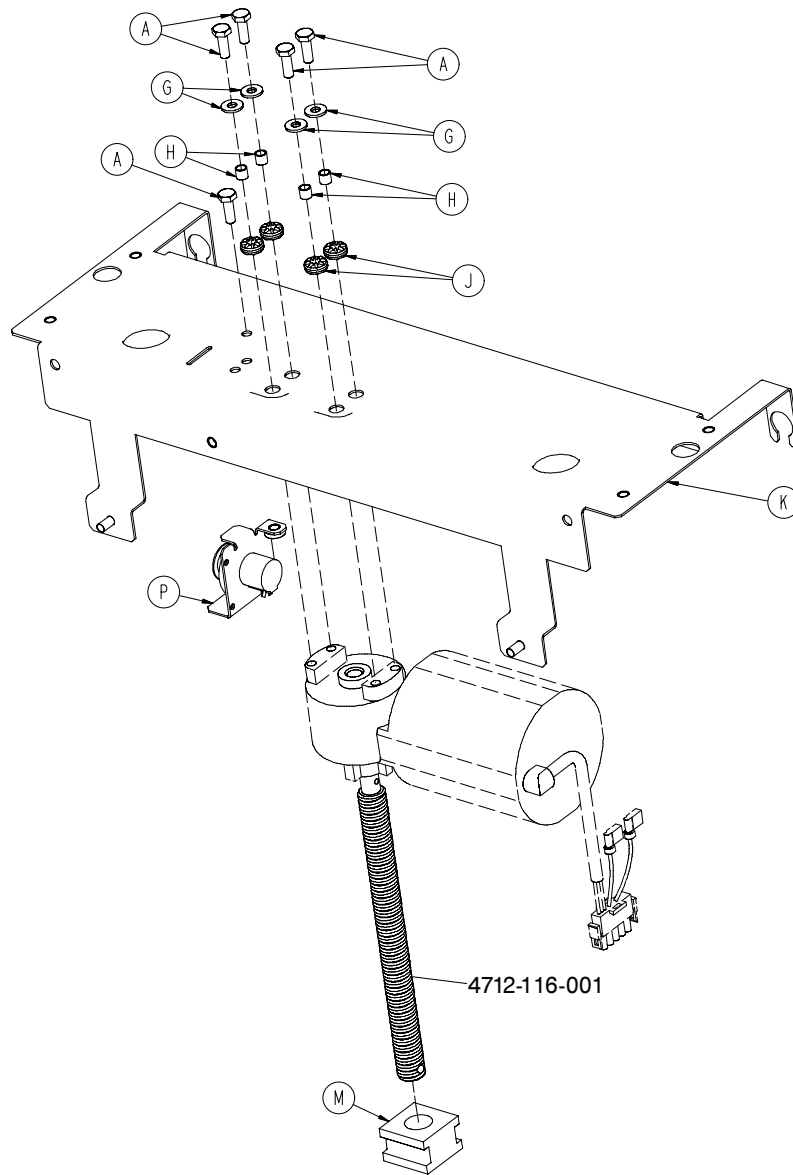



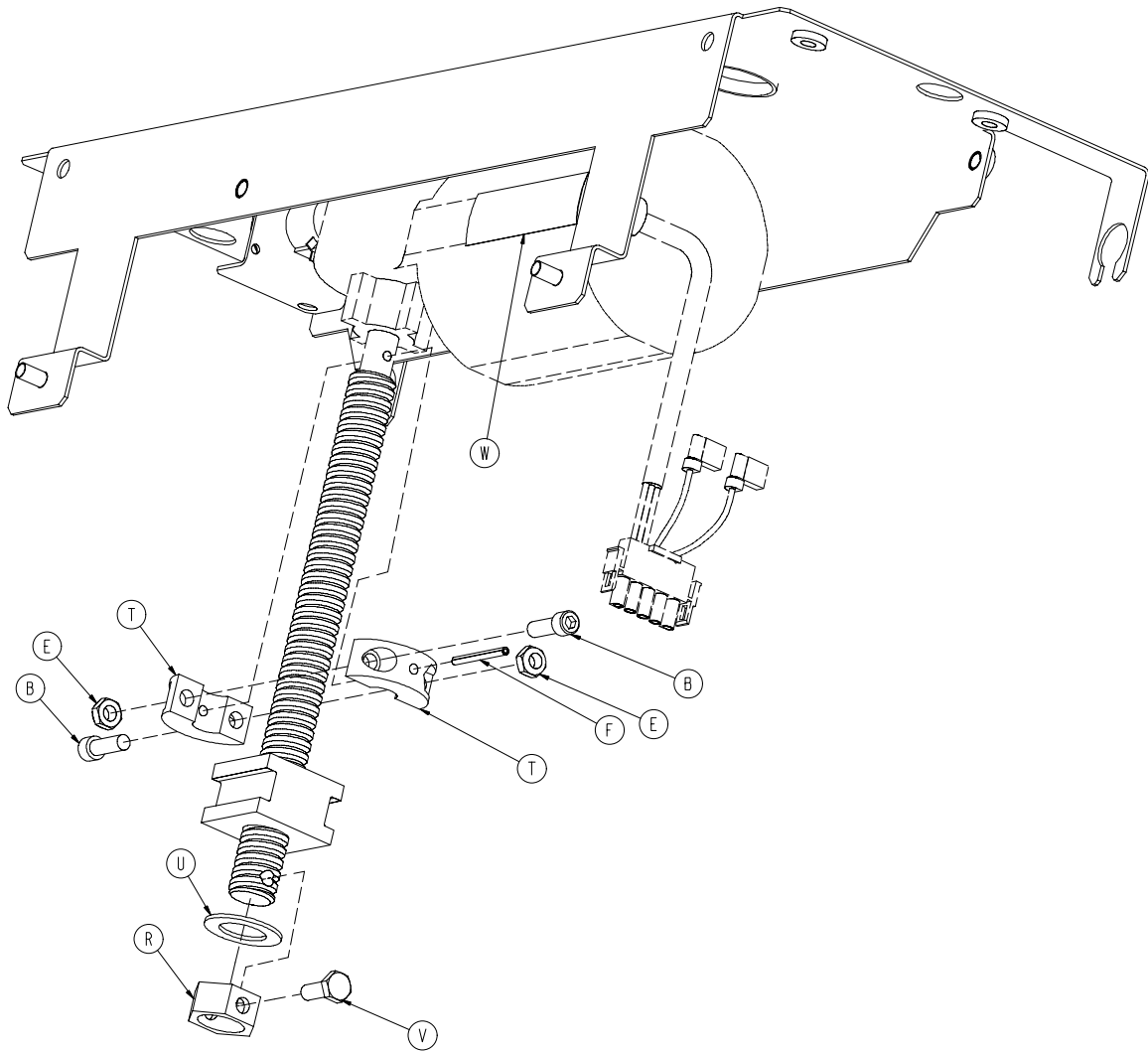



A	0003-214-000	5
B	0004-090-000	2
E	0016-102-000	2
F	0026-052-000	1
G	0011-063-000	4
H	0715-001-333	4
J	3000-200-228	4
K	4701-040-001	1
M	4701-040-124	1
P	4701-080-025	1
R	4701-040-157	1
T	5000-030-286	2
U	0011-148-000	1
V	0004-334-000	1
W	4701-090-056	1





Rev AA

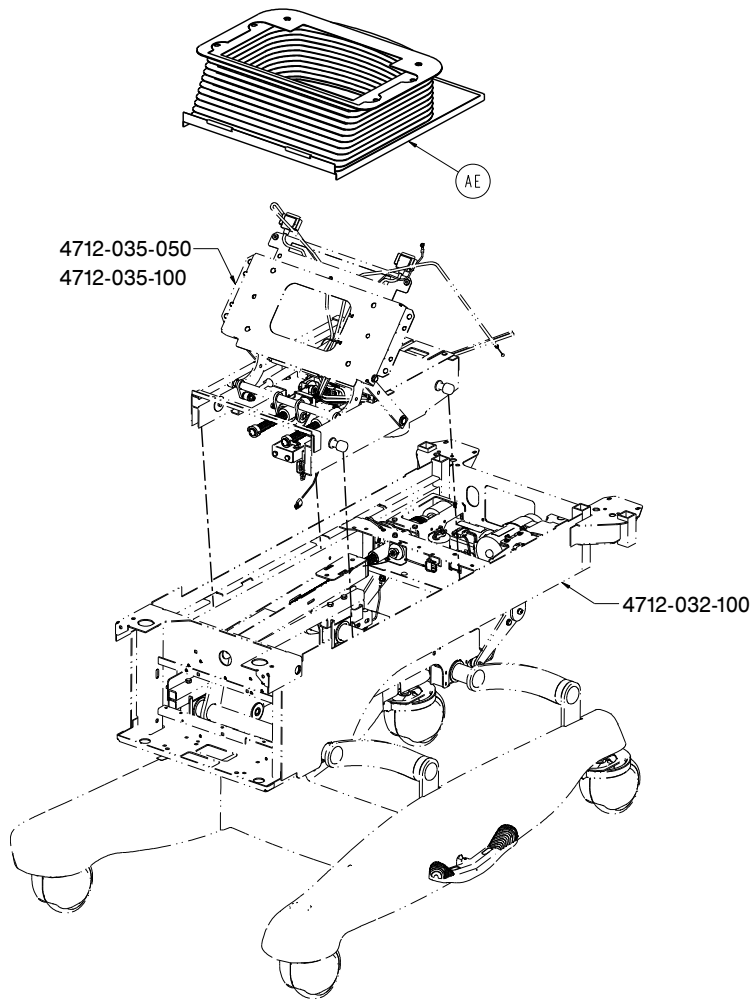



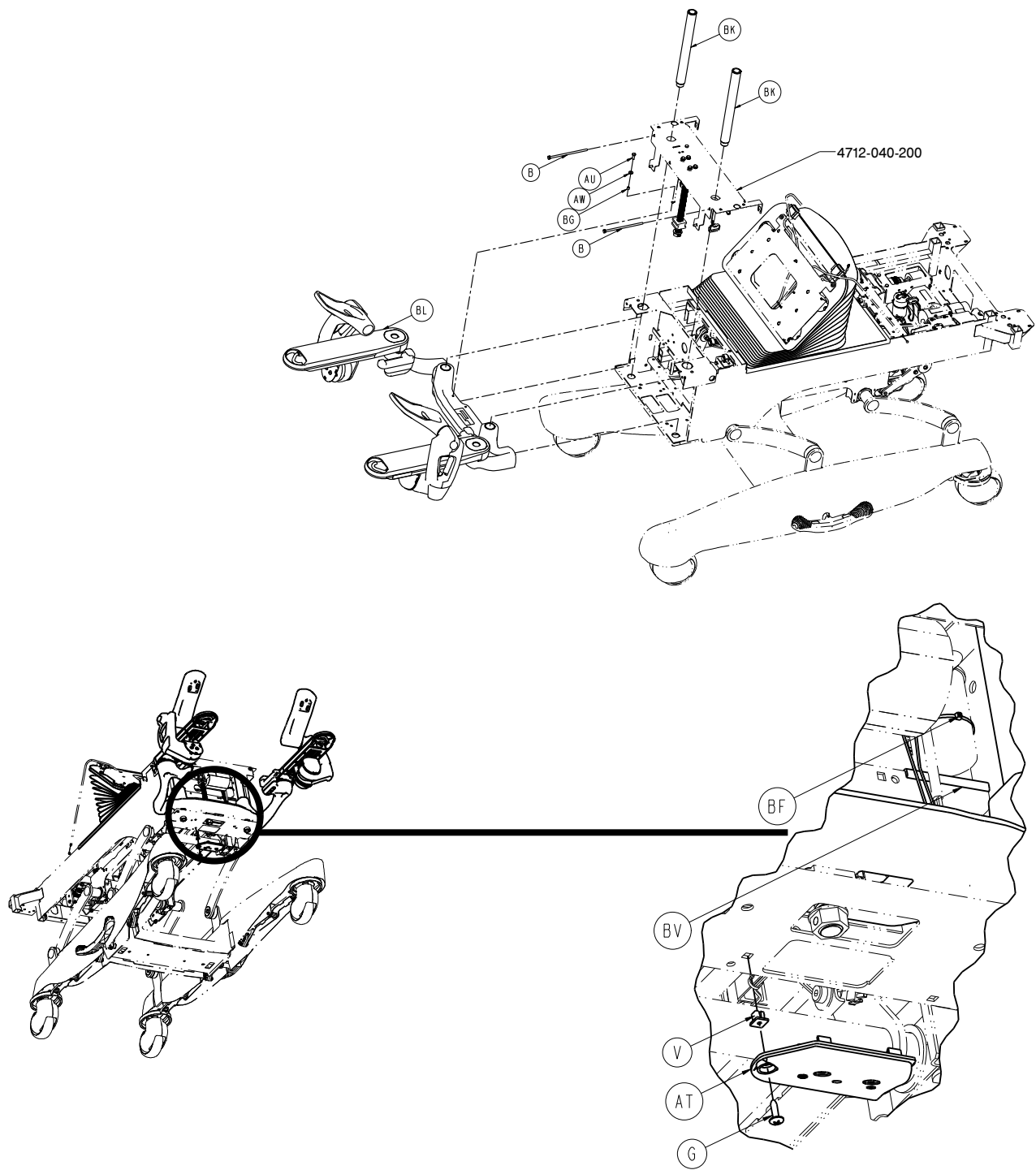


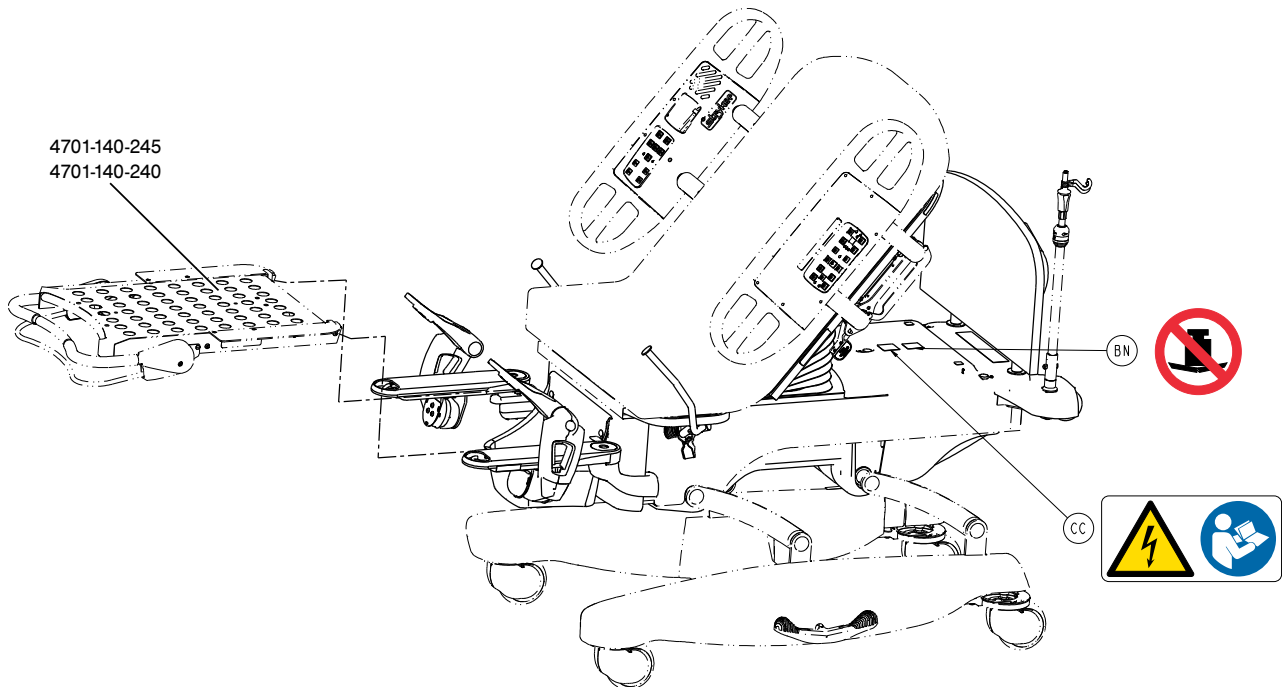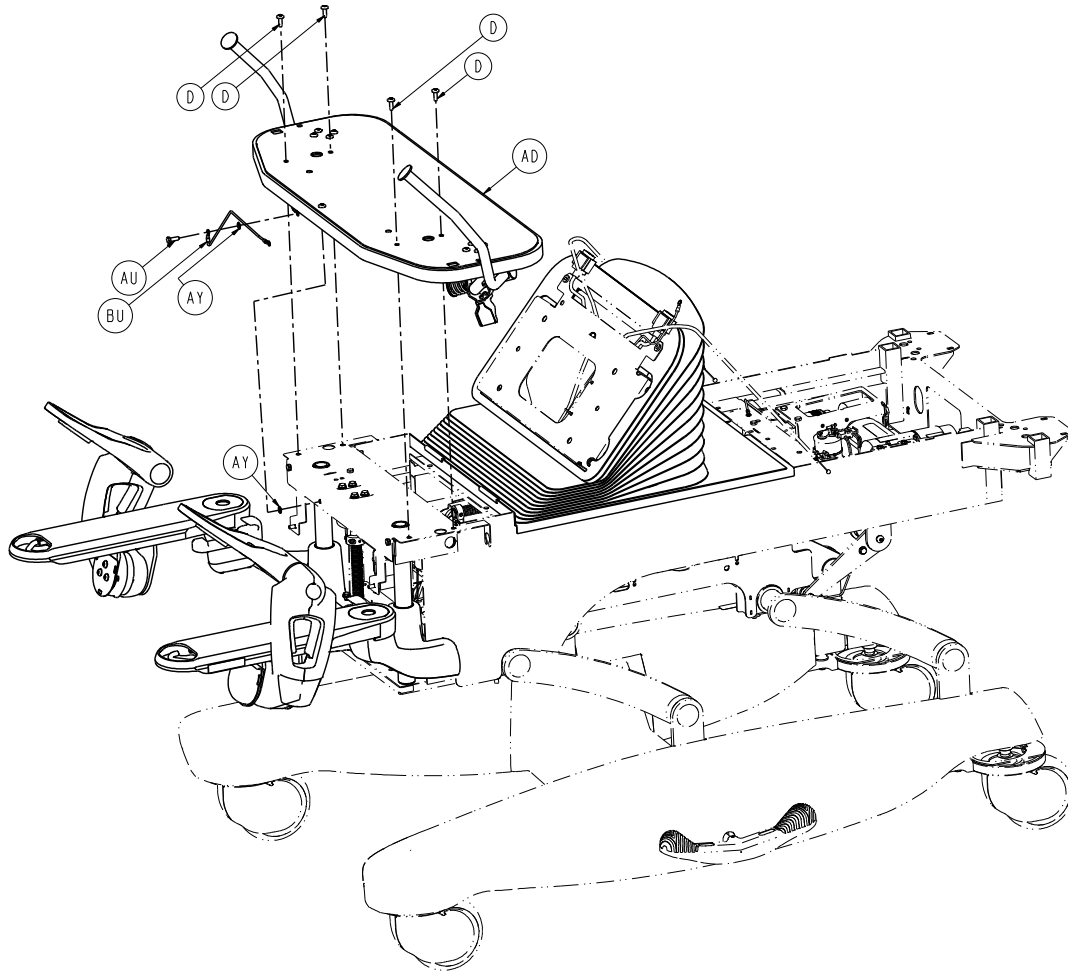



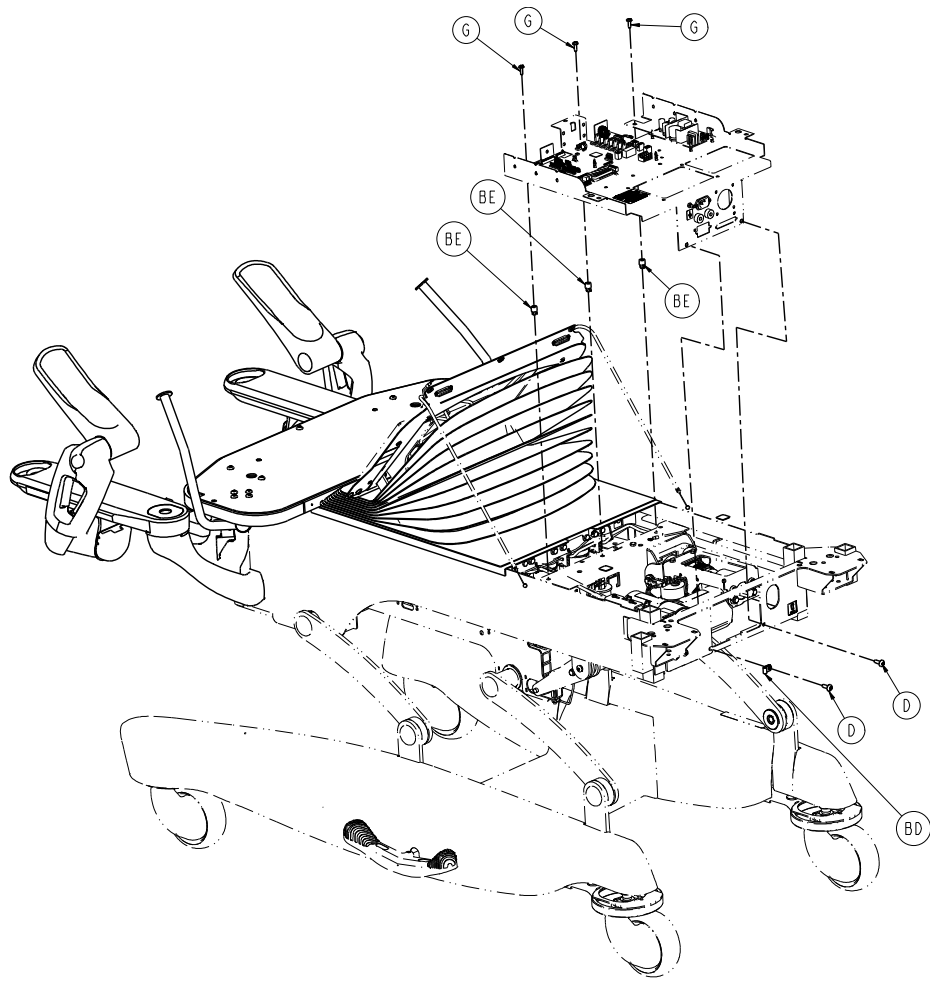


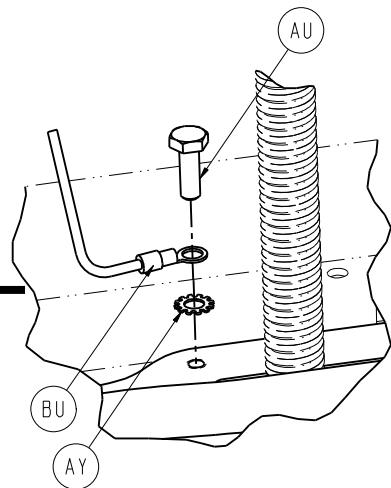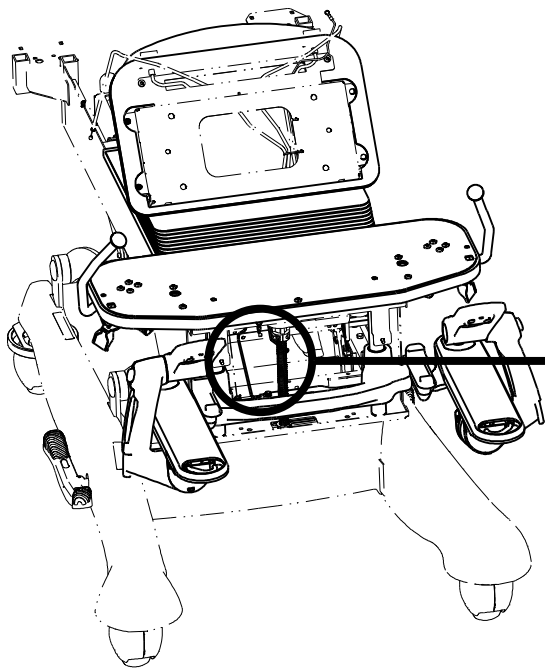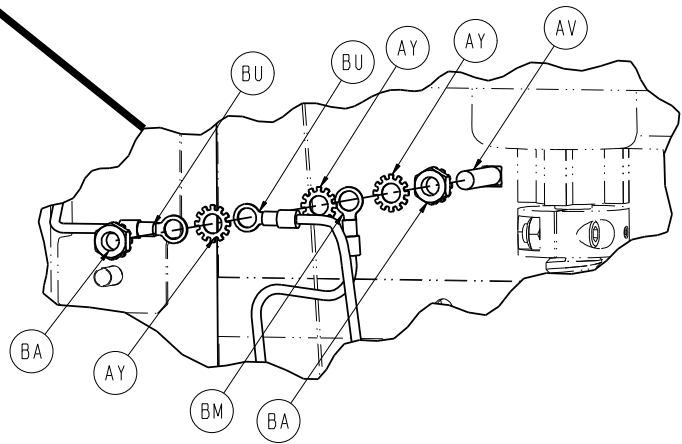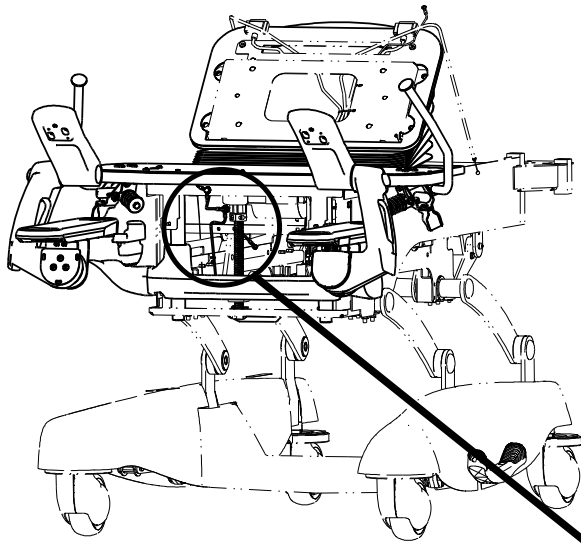



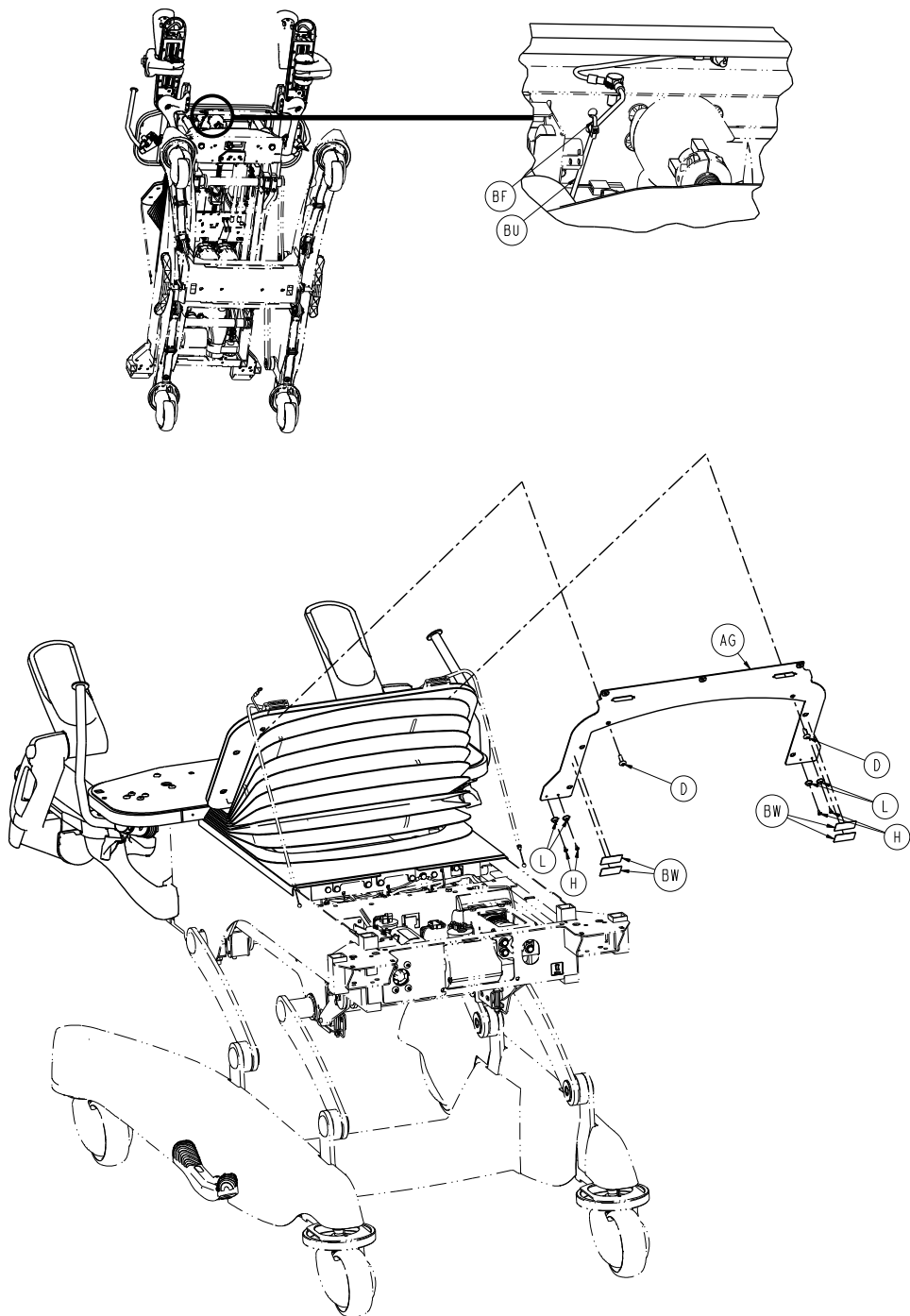


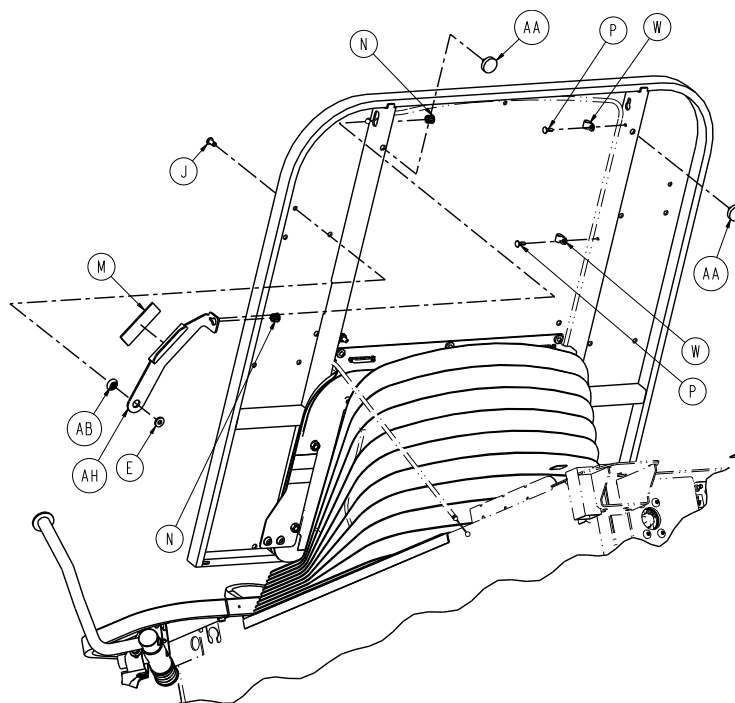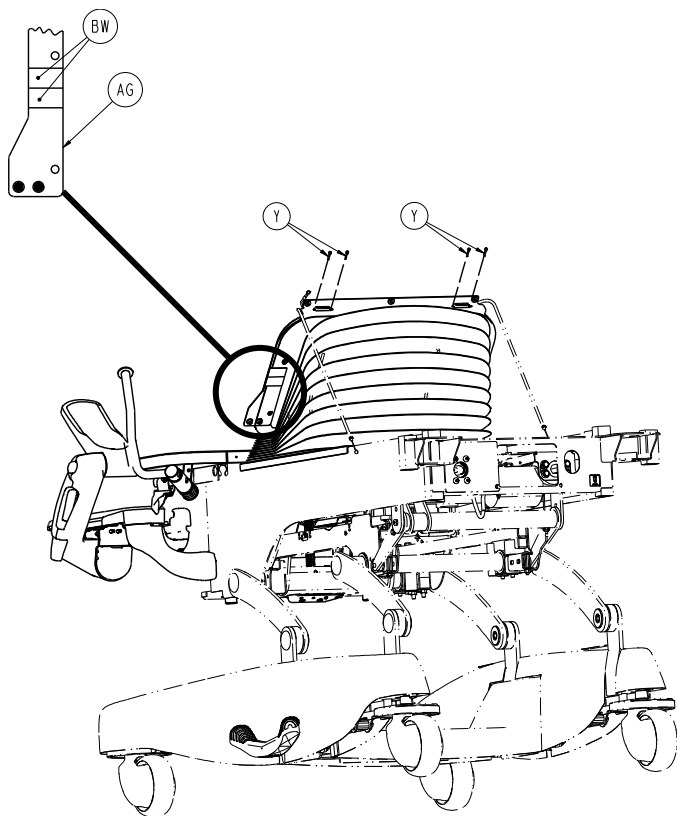


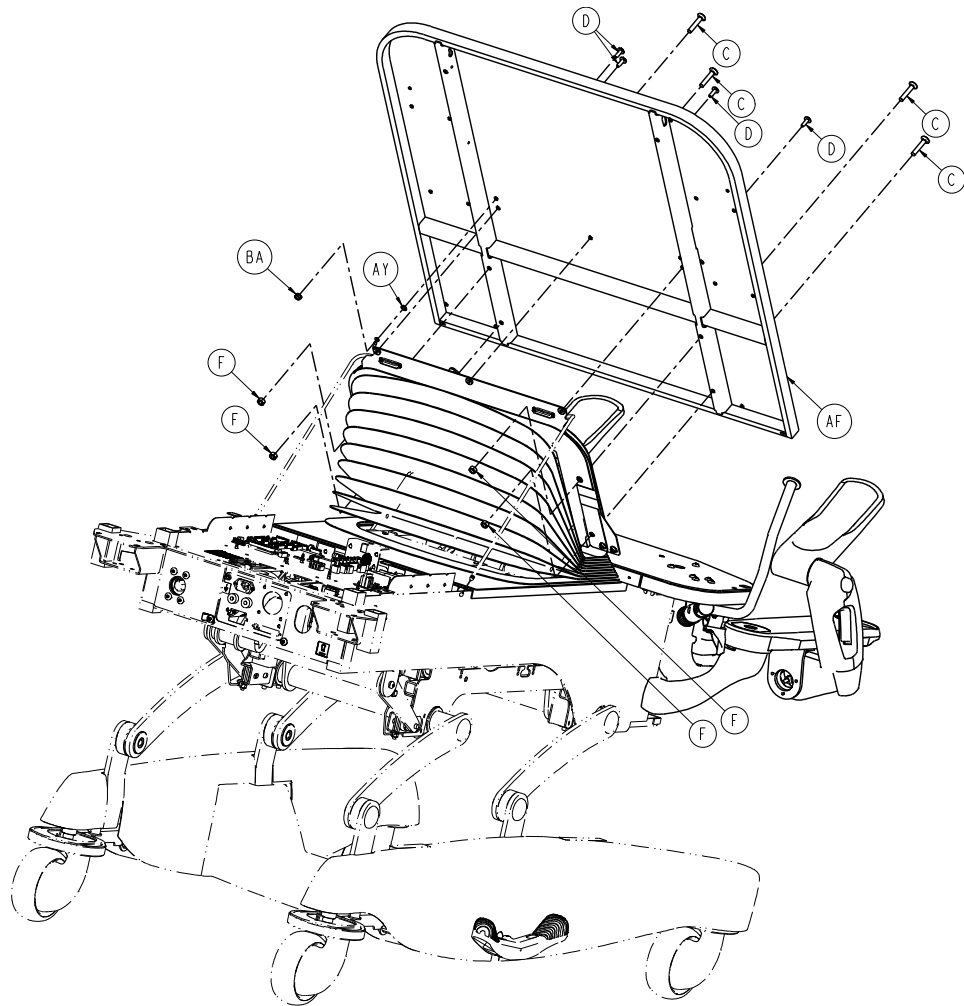


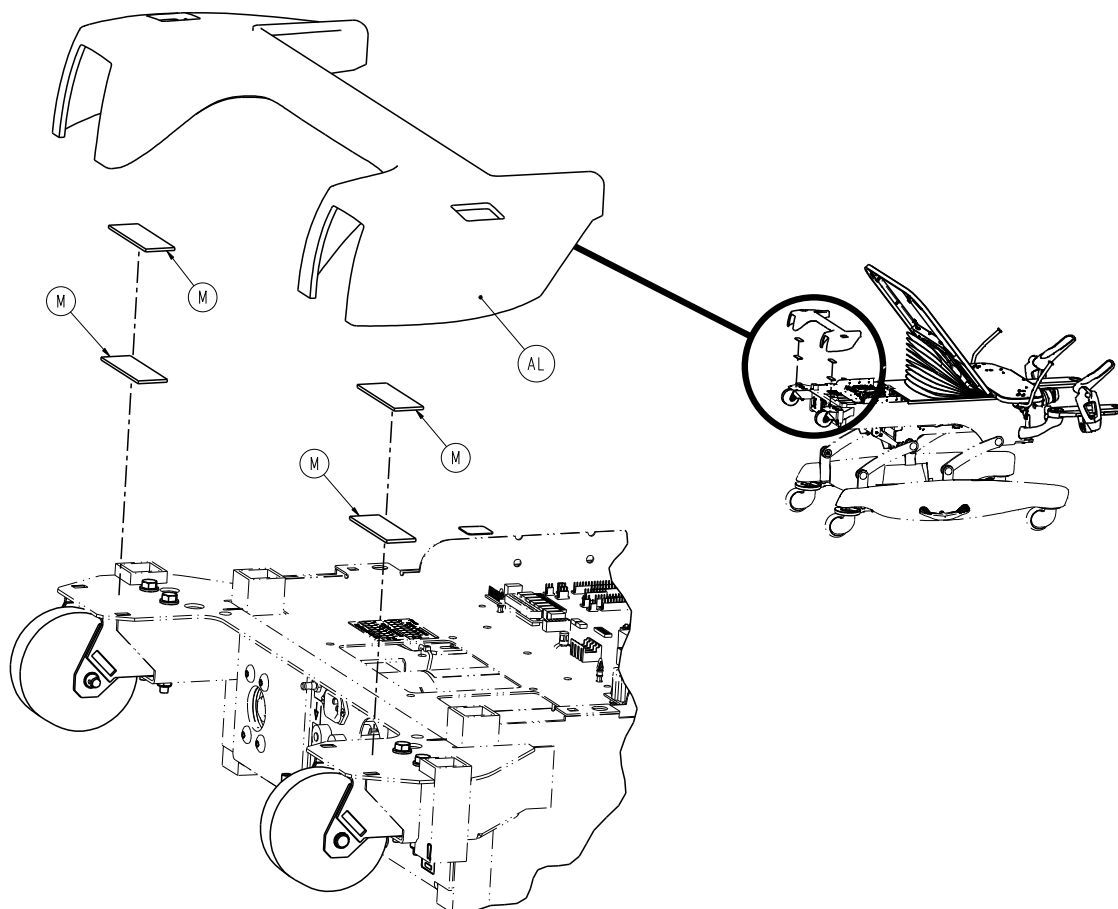
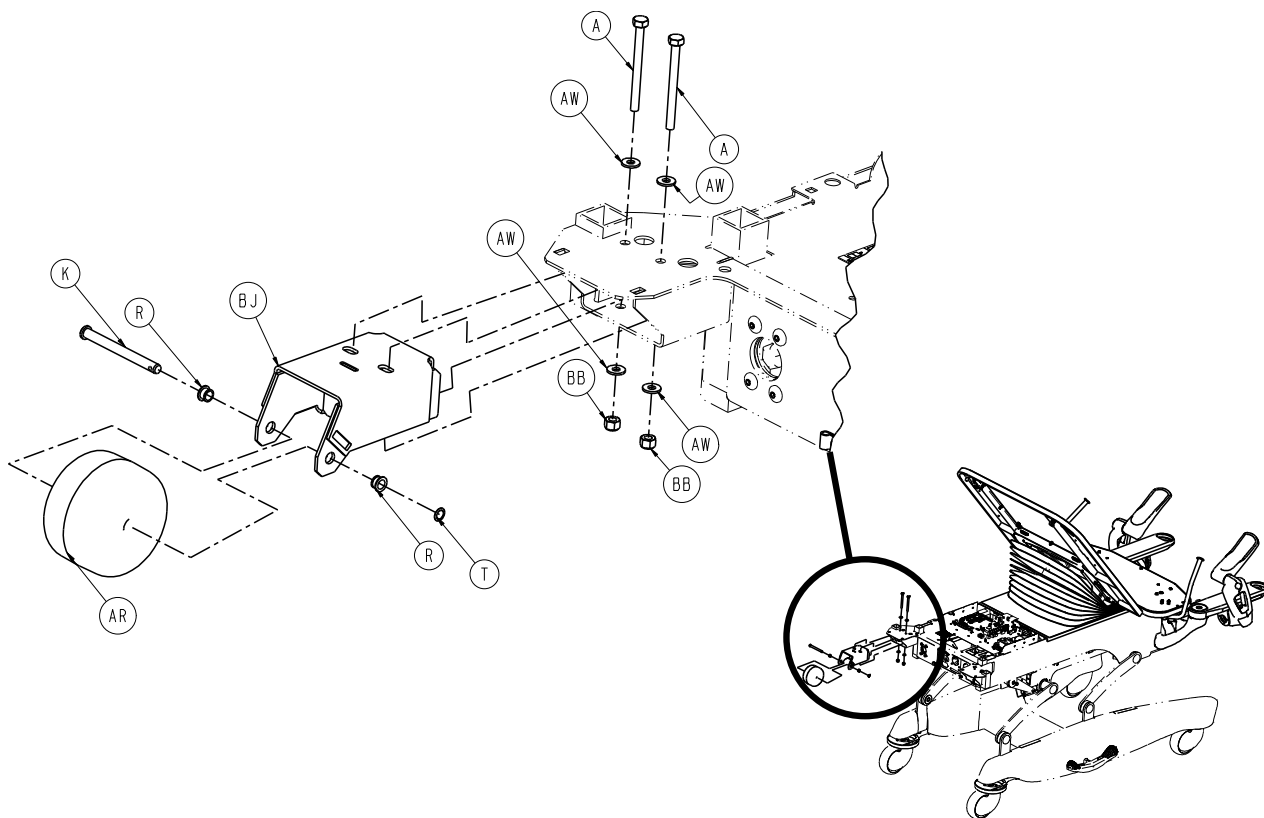


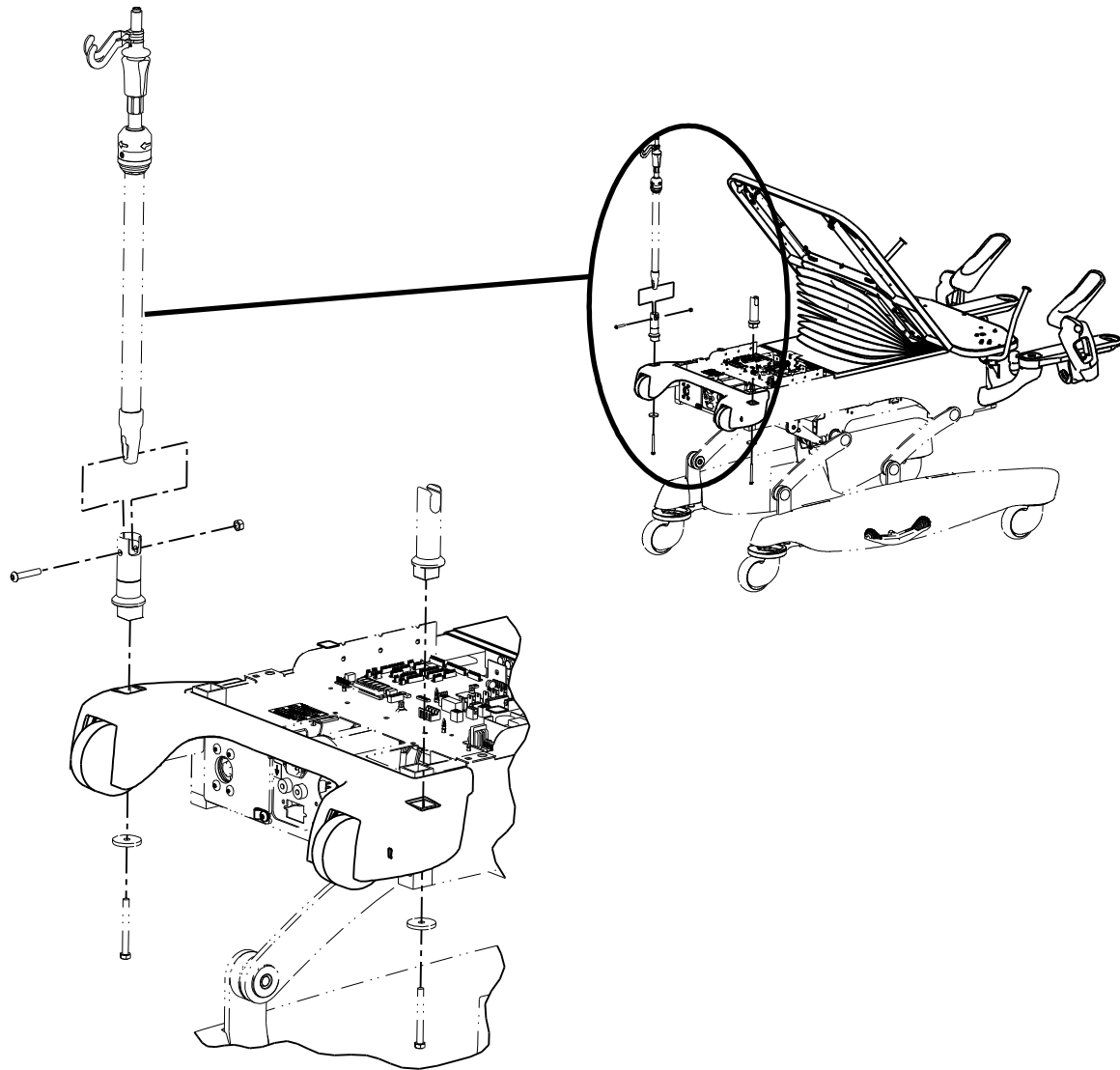

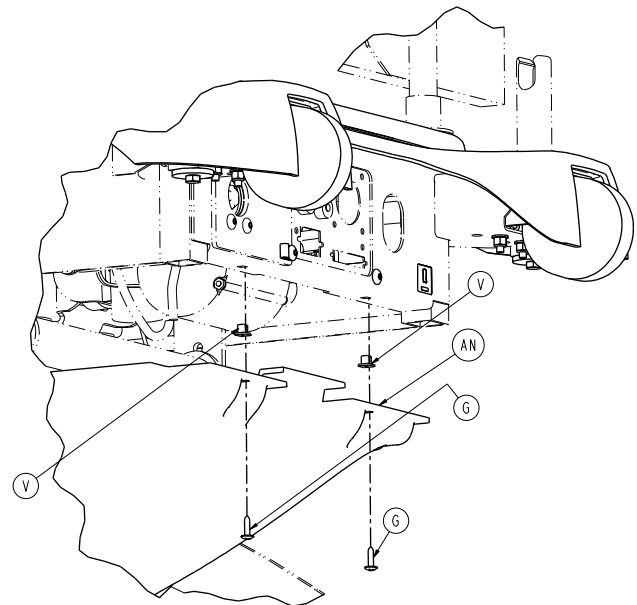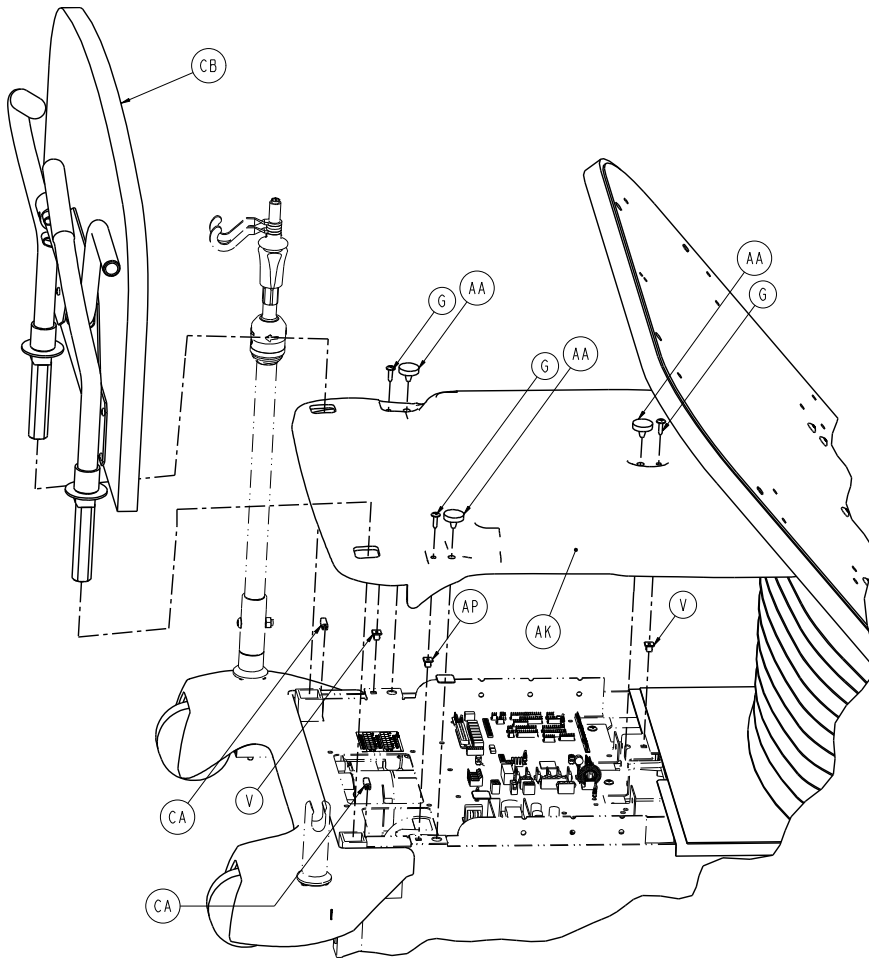

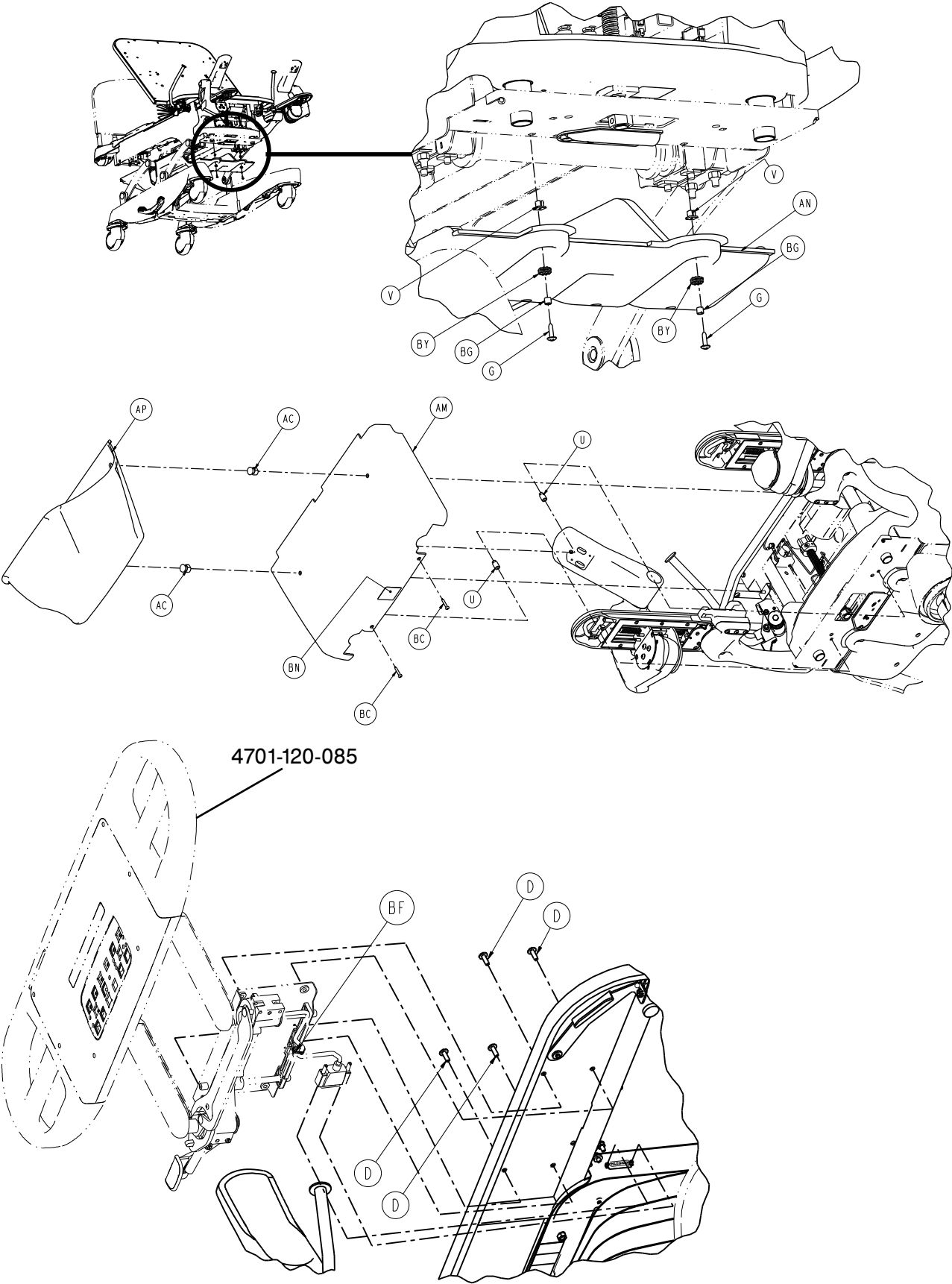

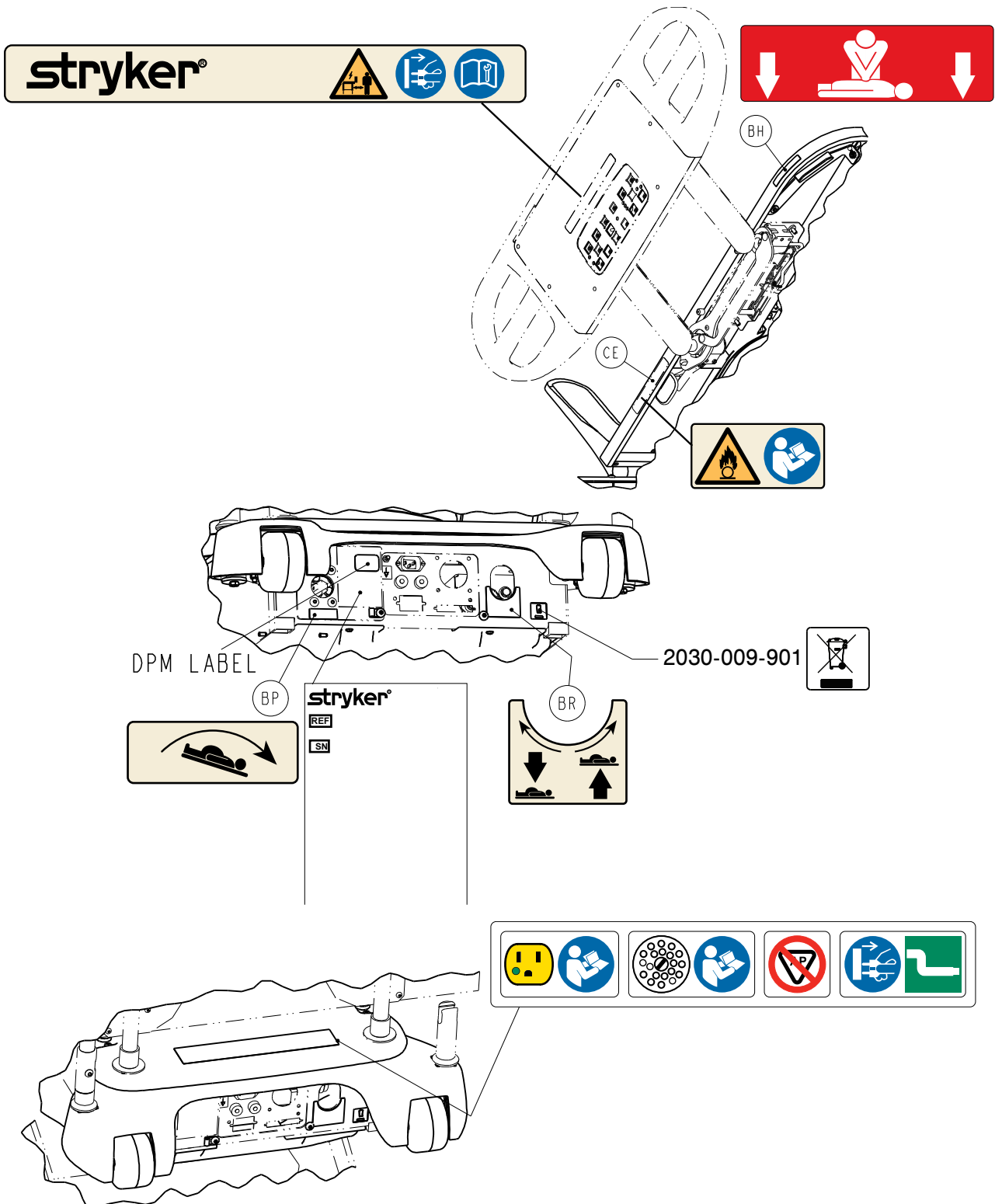

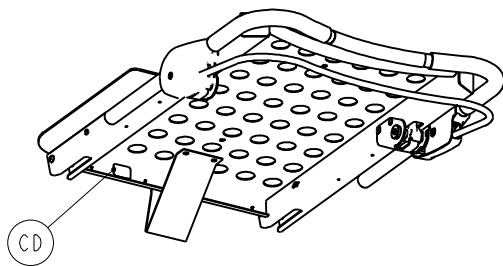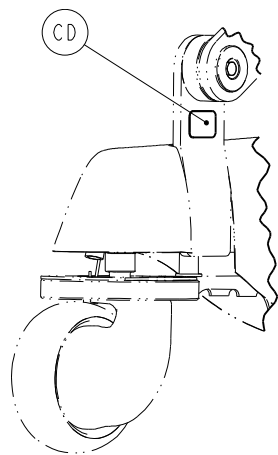

A	0003-056-000	4
B	0003-373-000	2
C	0005-040-000	4
D	0007-065-000	20
E	0011-376-000	2
F	0016-116-000	4
G	0023-080-000	11
H	0025-038-000	4
J	0025-120-000	2
K	0026-278-000	2
L	0029-003-000	4
M	0029-009-000	6
N	0030-052-000	4
P	0049-005-000	4
R	0052-245-000	4
T	0052-727-000	2
U	0052-800-000	2
V	3000-300-002	8
W	3000-300-478	4
Y	3001-300-007	4
AA	3001-300-008	5


4701-130-000

AB	3001-300-099	2
AC	4701-030-105	2
AD	4701-130-110	1
AE	4701-030-130	1
AF	4701-030-150	1
AG	4701-030-158	1
AH	4701-230-164	1
AJ	4701-230-165	1
AK	4701-130-172	1
AL	4701-130-173	1
AM	4701-030-174	1
AN	4701-130-175	1
AP	4701-036-005	1
AR	3000-335-011	2
AT	4701-036-201	1
AU	0003-214-000	3
AV	0005-030-000	1
AW	0011-063-000	9
AY	0013-010-000	7
BA	0016-006-000	3
BB	0016-028-000	4
BC	0023-309-000	2
BD	0034-020-000	1
BE	0052-907-000	3
BF	0059-086-000	4
BG	0715-001-333	3
BH	4701-109-963	2
BJ	4701-030-210	2
BK	4701-040-041	2
BL	4701-140-225	1
BM	4701-080-018	1
BN	4701-109-959	2
BP	4701-090-046	1
BR	4701-090-047	1
BU	3002-300-870	2
BV	0058-056-000	6"
BW	0044-032-000	4
BY	3000-200-228	2
CA	4701-030-117	2

4701-130-000

CB	4701-130-125	1
CC	4701-109-958	1
CE	4701-109-954	2

Rev A

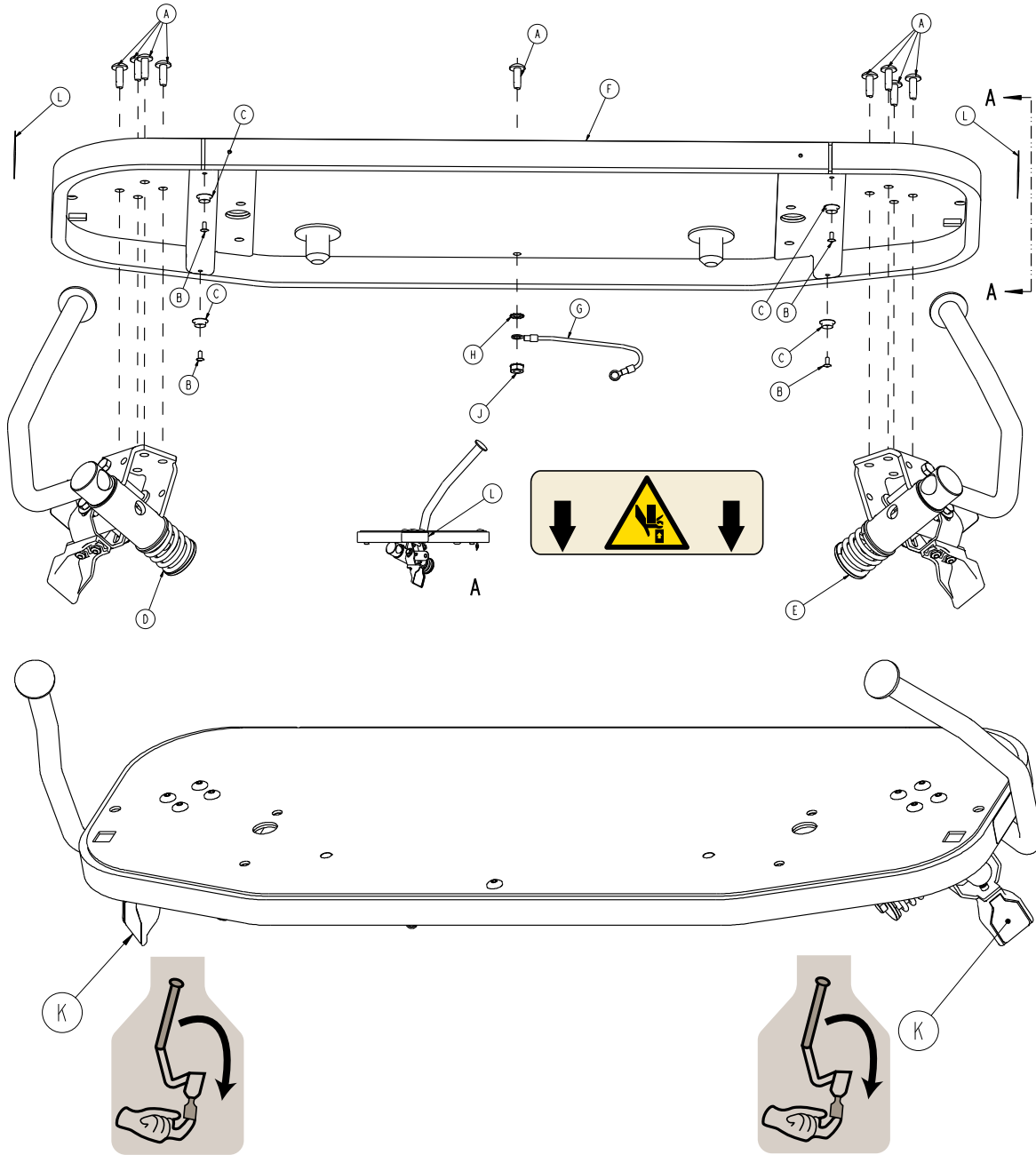


A	0007-065-000	9
B	0025-038-000	4
C	0029-003-000	4
D	5010-330-061	1
E	5010-330-060	1
F	4701-030-100	1
G	3002-300-870	1

4701-130-110


H	0013-010-000	1
J	0016-006-000	1
K	4701-109-972	2
L	4701-109-964	2

Rev A

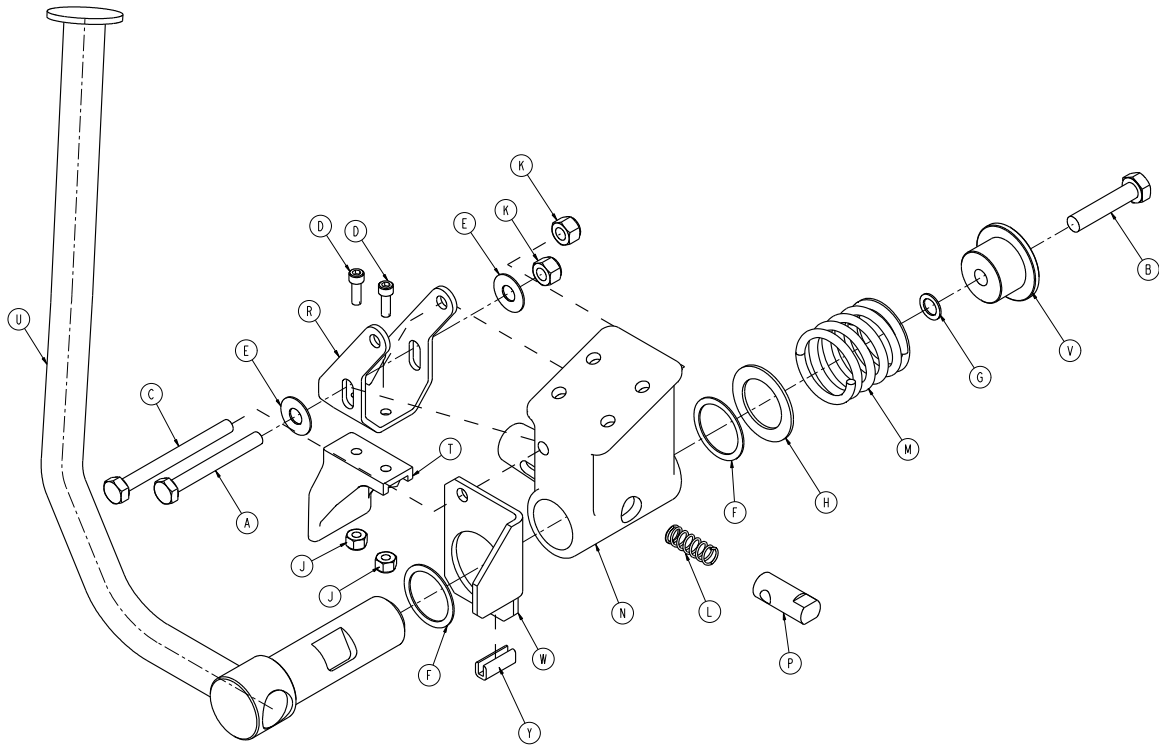

A	0003-032-000	1
B	0003-057-000	1
C	0003-073-000	1
D	0004-005-000	2
E	0011-179-000	2
F	0011-392-000	2
G	0011-397-000	1
H	0011-406-000	1
J	0016-014-000	2
K	0016-028-000	2
L	0038-344-000	1
M	0038-391-000	1
N	5000-030-125	1
P	5000-030-128	1
R	5000-030-129	1
T	5000-040-118	1
U	5010-230-063	1
V	5000-030-315	1
W	4701-036-016	1
Y	4701-030-117	1


Rev A

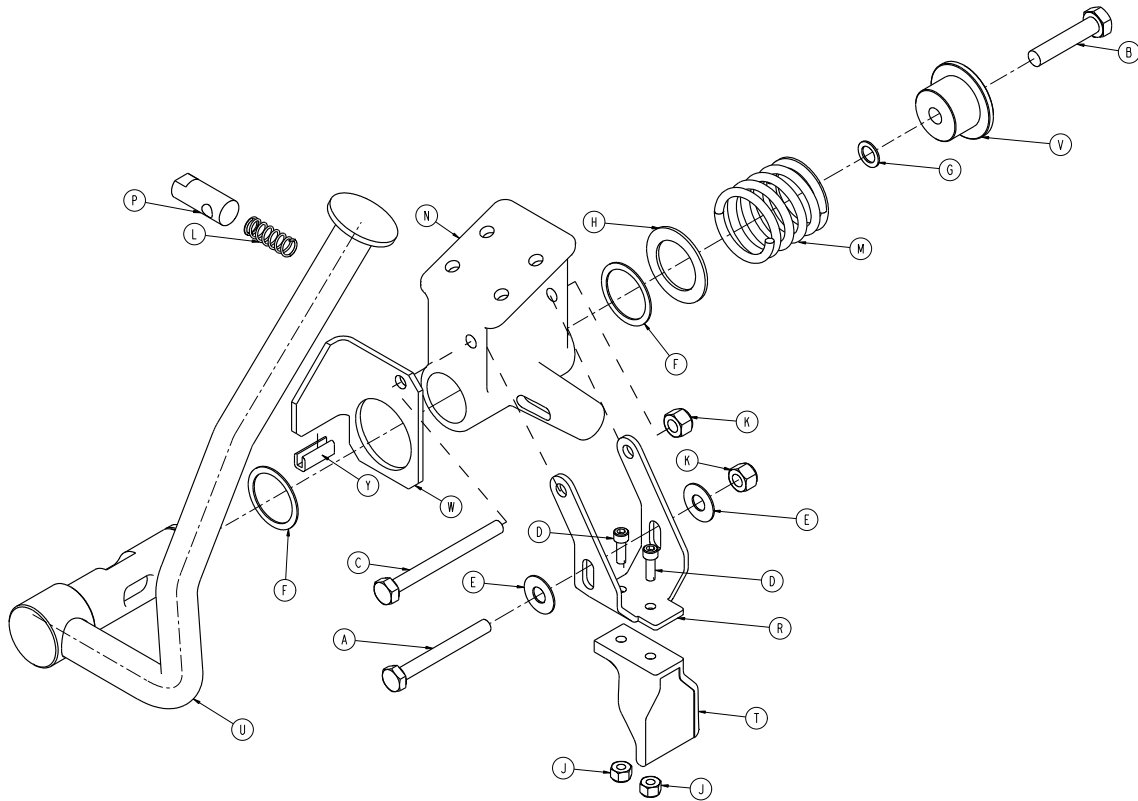

A	0003-079-000	1
B	0003-057-000	1
C	0003-073-000	1
D	0004-005-000	2
E	0011-179-000	2
F	0011-392-000	2
G	0011-397-000	1
H	0011-406-000	1
J	0016-014-000	2
K	0016-028-000	2
L	0038-344-000	1
M	0038-391-000	1
N	5000-030-124	1
P	5000-030-128	1
R	5000-030-129	1
T	5000-040-118	1
U	5010-230-062	1
V	5000-030-315	1


W	4701-036-017	1
Y	4701-030-117	1


Rev A

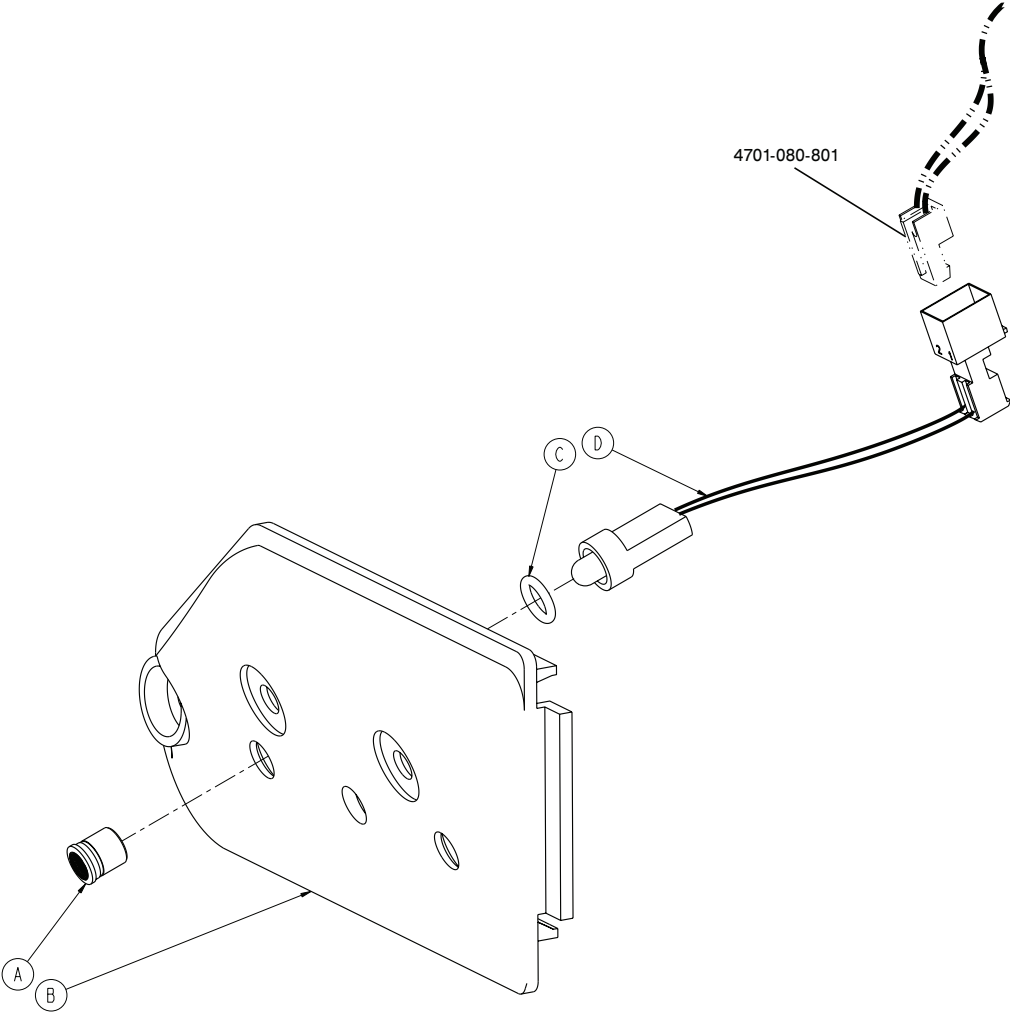



A	3002-310-010	1
B	3002-310-018	1
C	3002-310-025	1
D	4701-036-801	1

Rev A

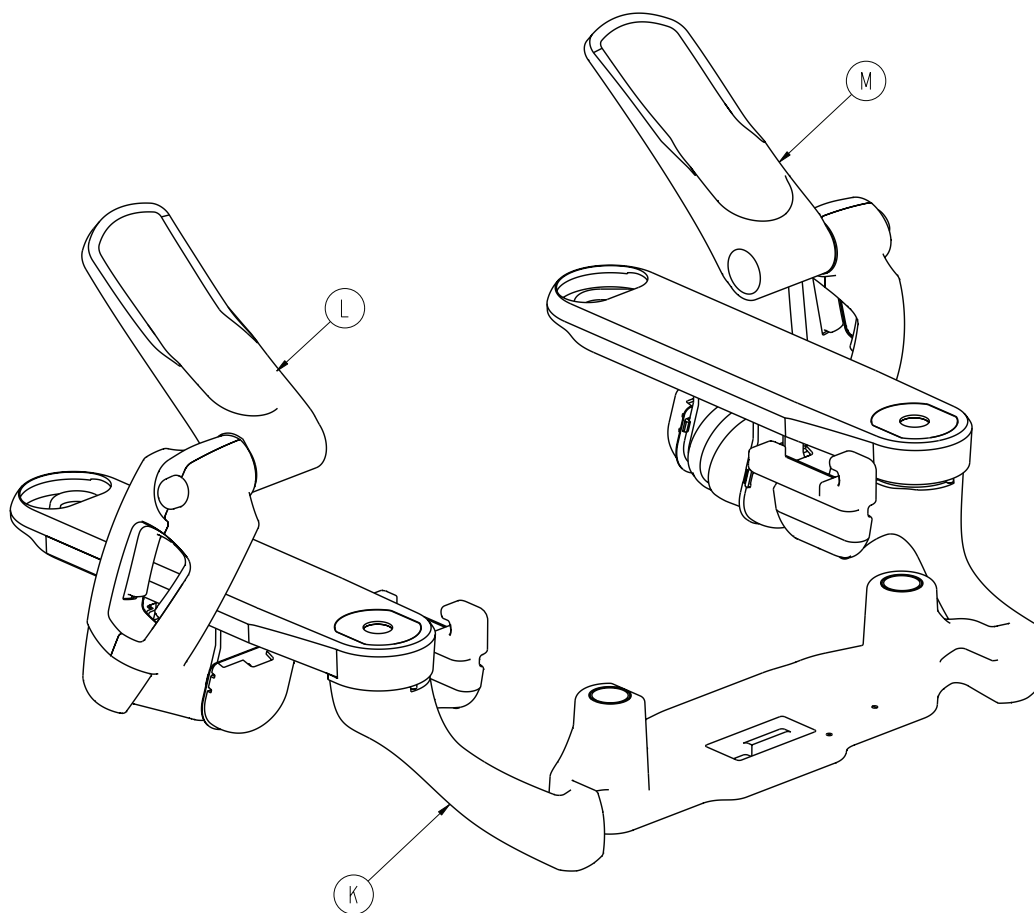

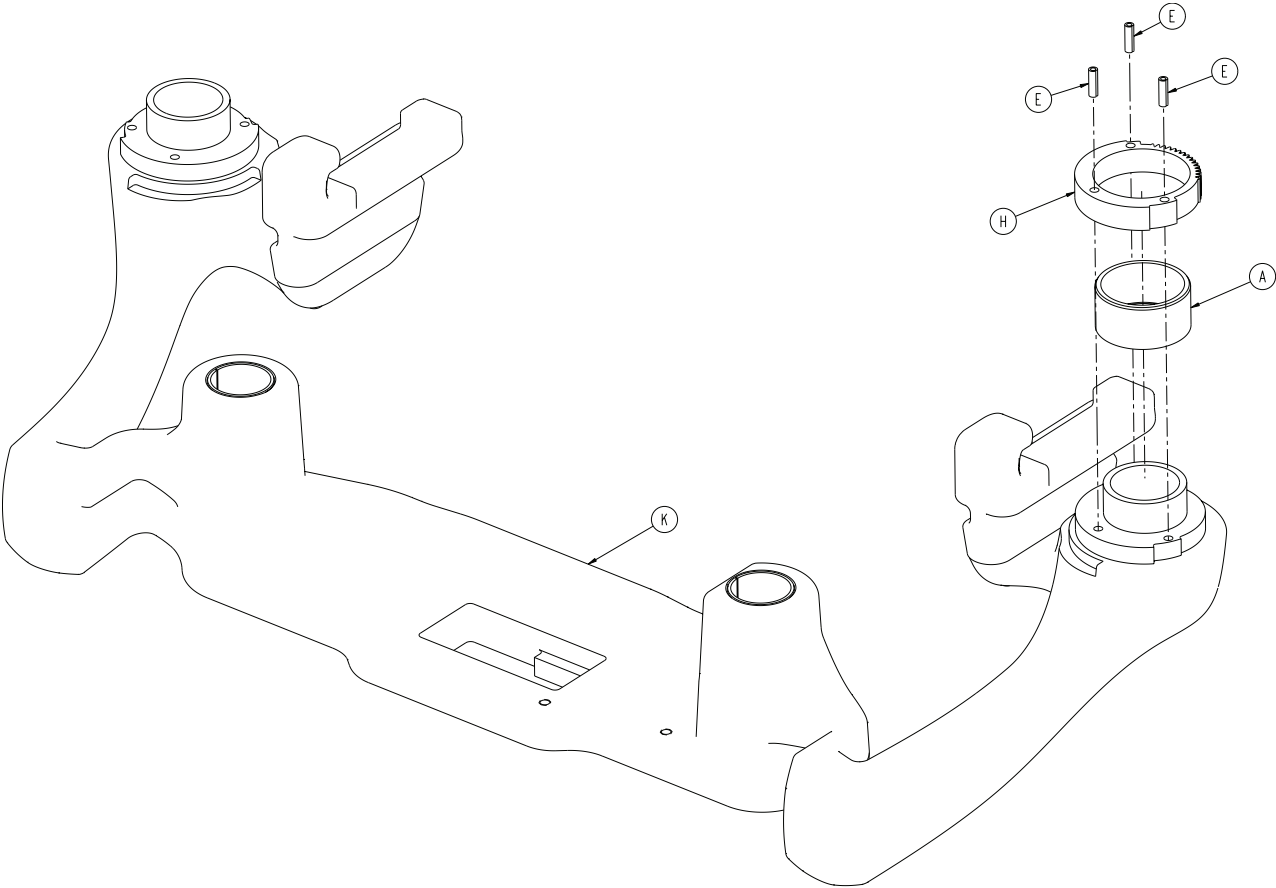


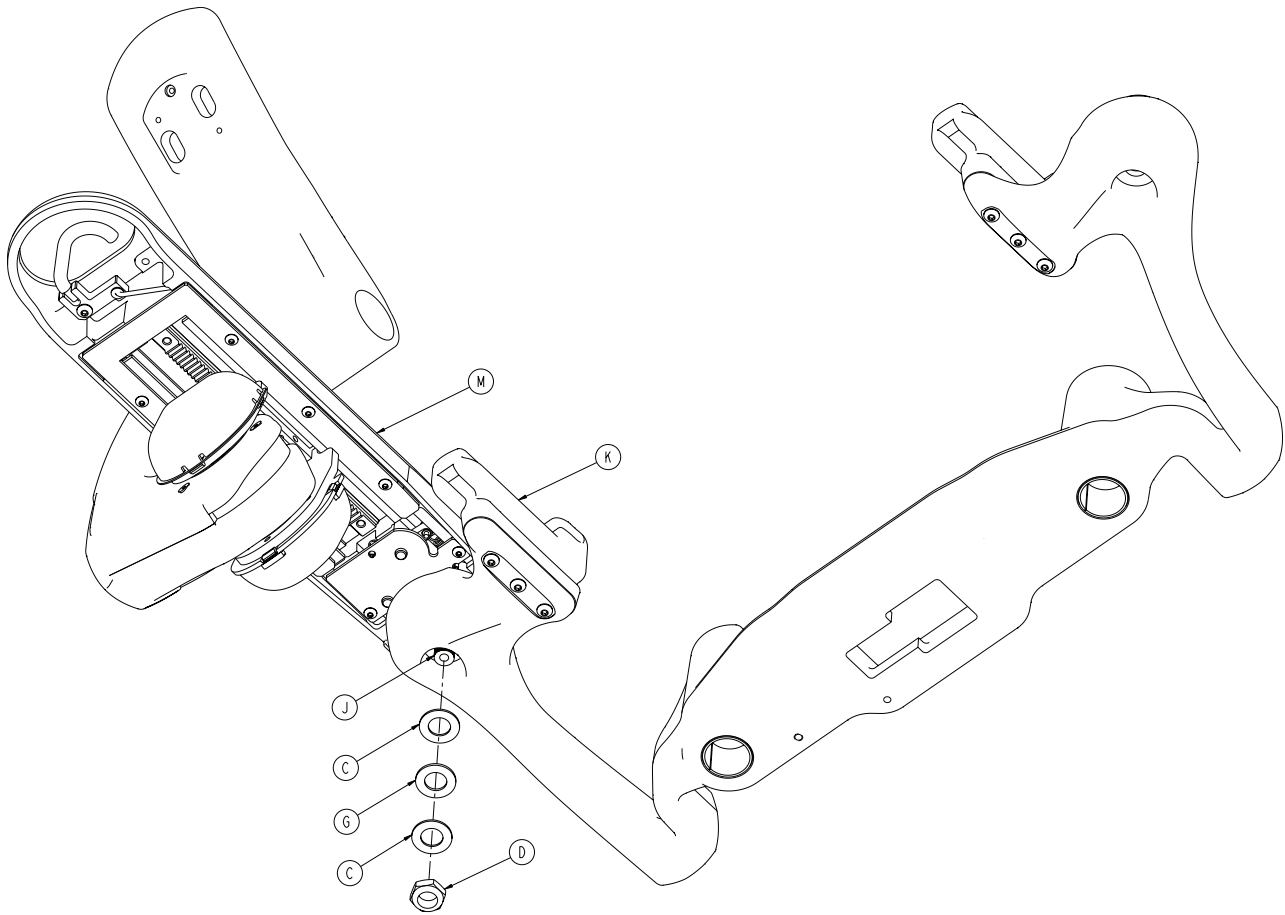



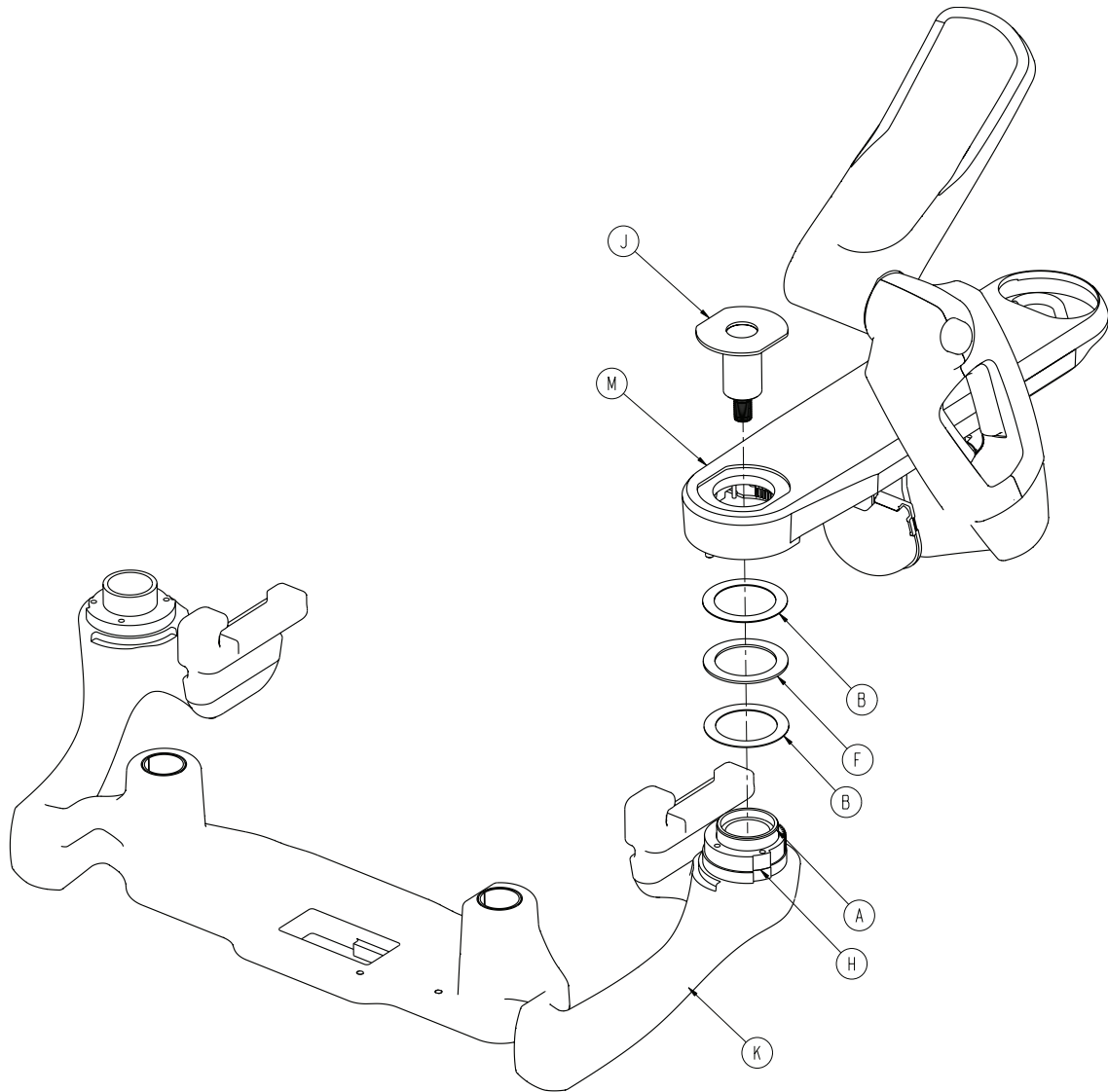


A	0011-535-000	2
B	0011-537-000	4
C	0014-083-000	4
D	0016-031-000	2
E	0026-172-000	6
F	0081-635-000	2
G	3000-300-466	2
H	4701-040-037	2
J	4701-040-052	2
K	4701-040-205	1
L	4701-140-215	1
M	4701-140-220	1

Rev A

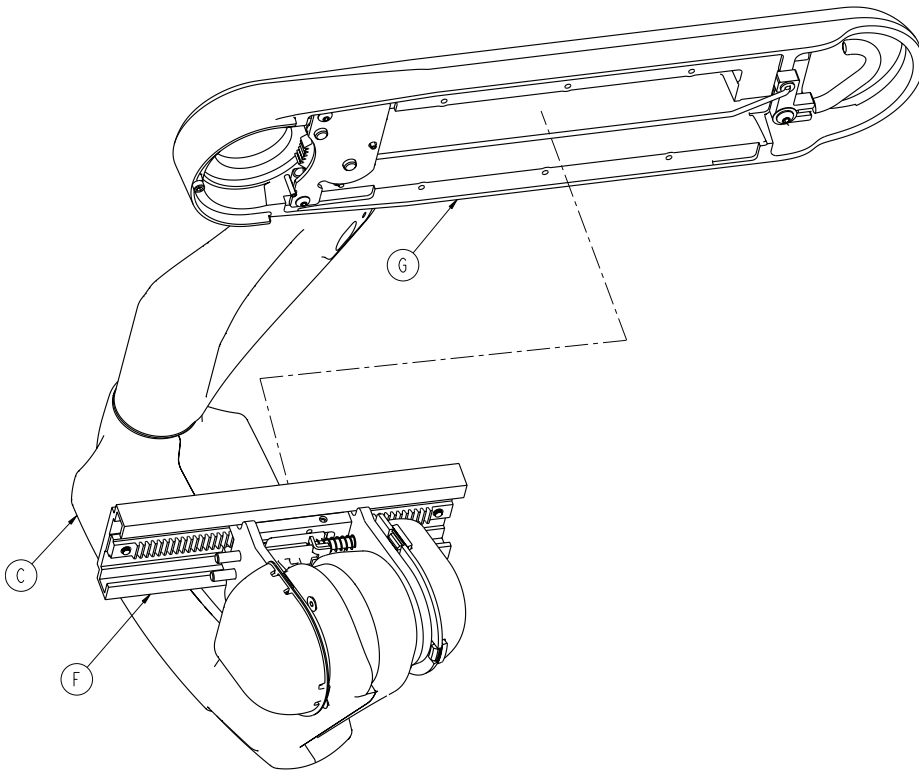

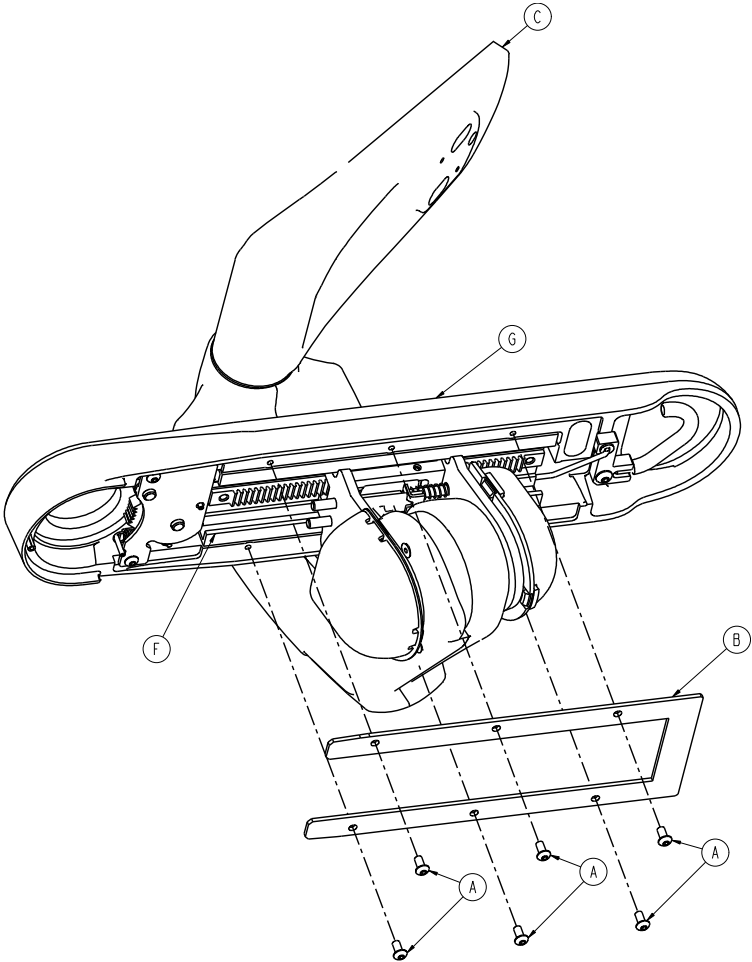


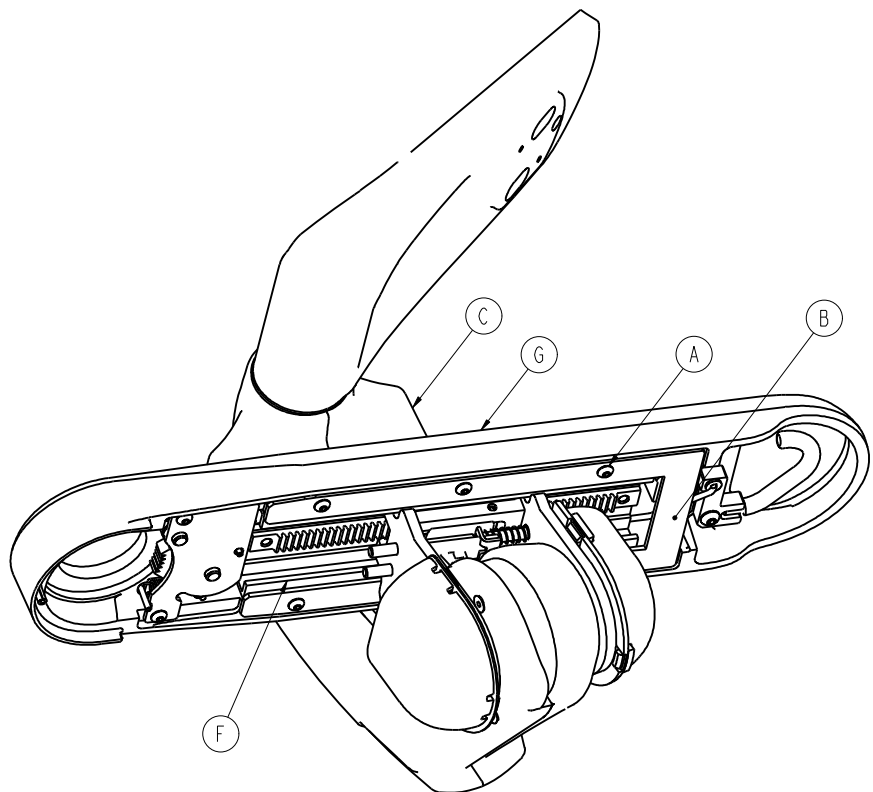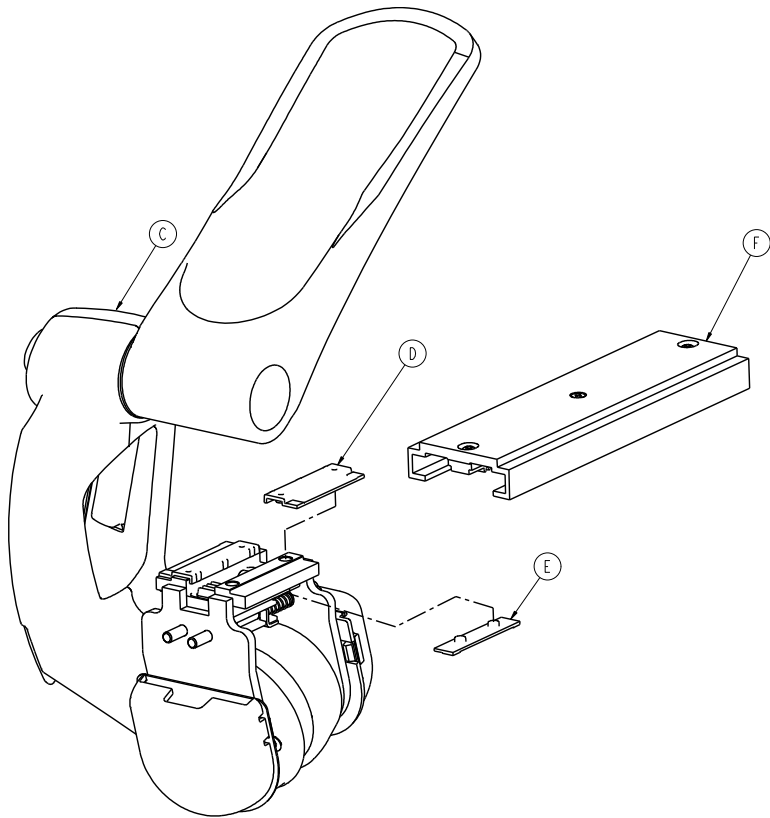


4701-140-215

A	0004-375-000	6
B	4701-040-051	1
C	4701-040-070	1
D	4701-040-106	2
E	4701-040-107	2
F	4701-040-130	1
G	4701-140-165	1

Rev A

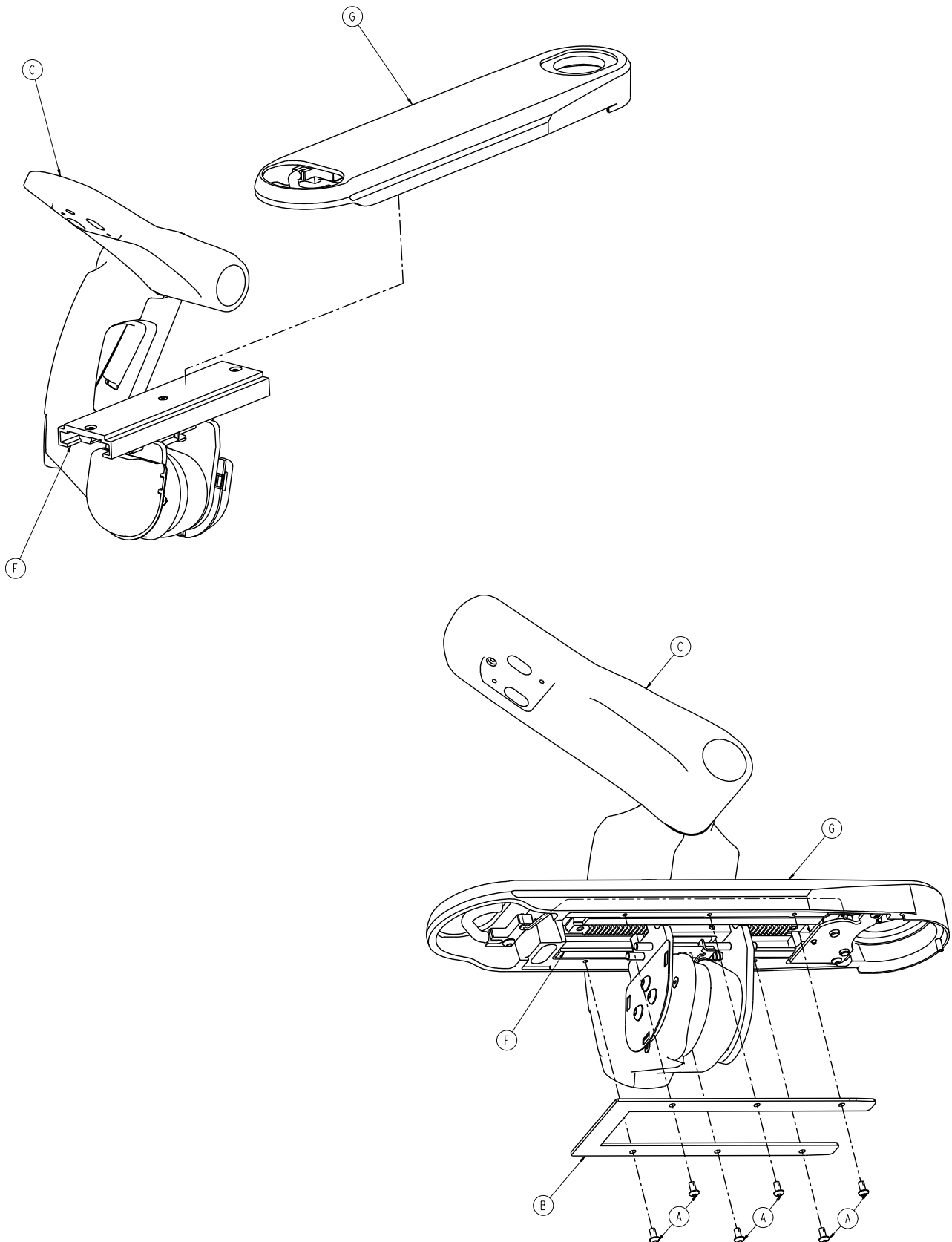

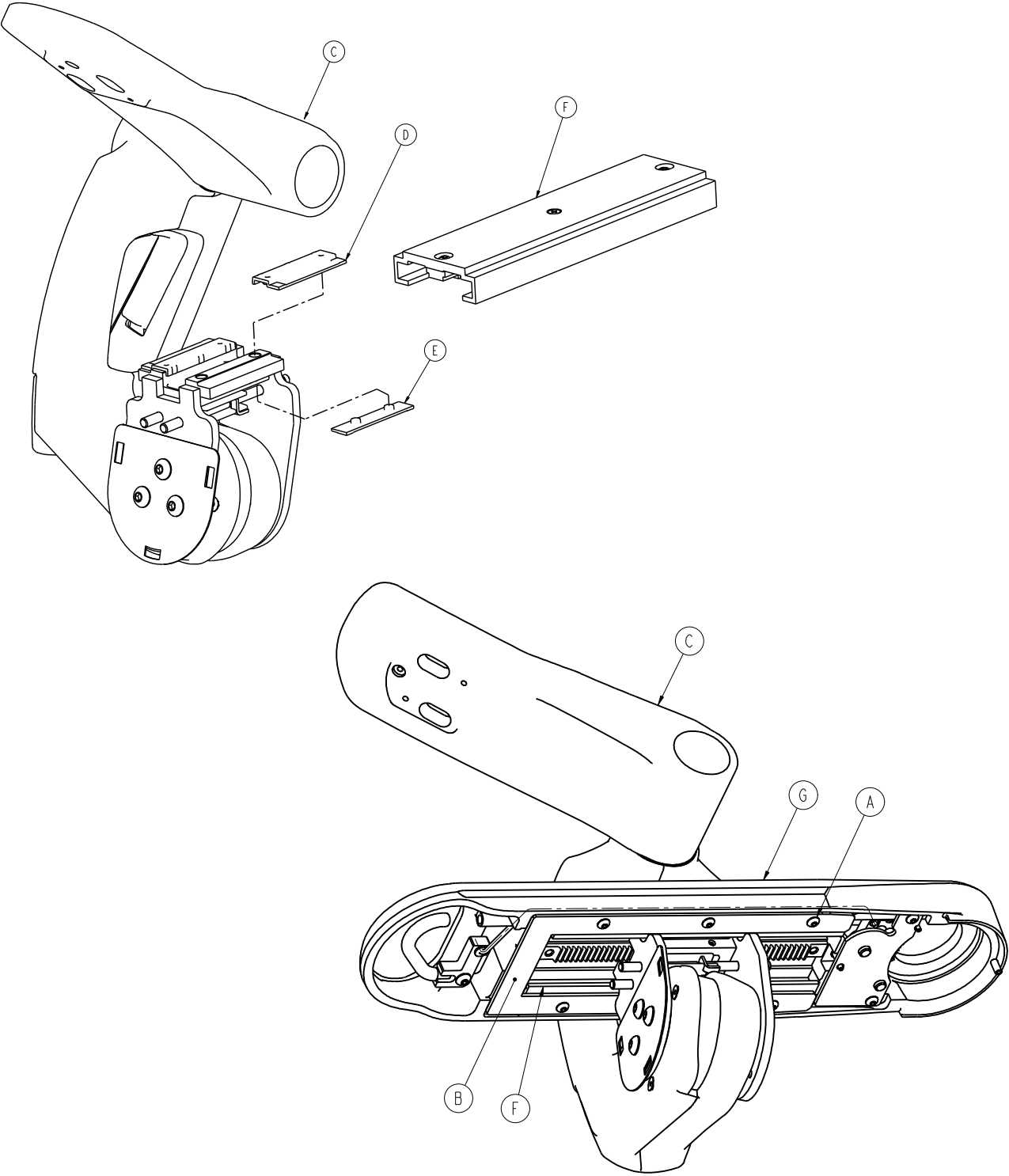


A	0004-375-000	6
B	4701-040-051	1
C	4701-040-060	1

4701-140-220


D	4701-040-106	2
E	4701-040-107	2
F	4701-040-130	1
G	4701-140-170	1


Rev B

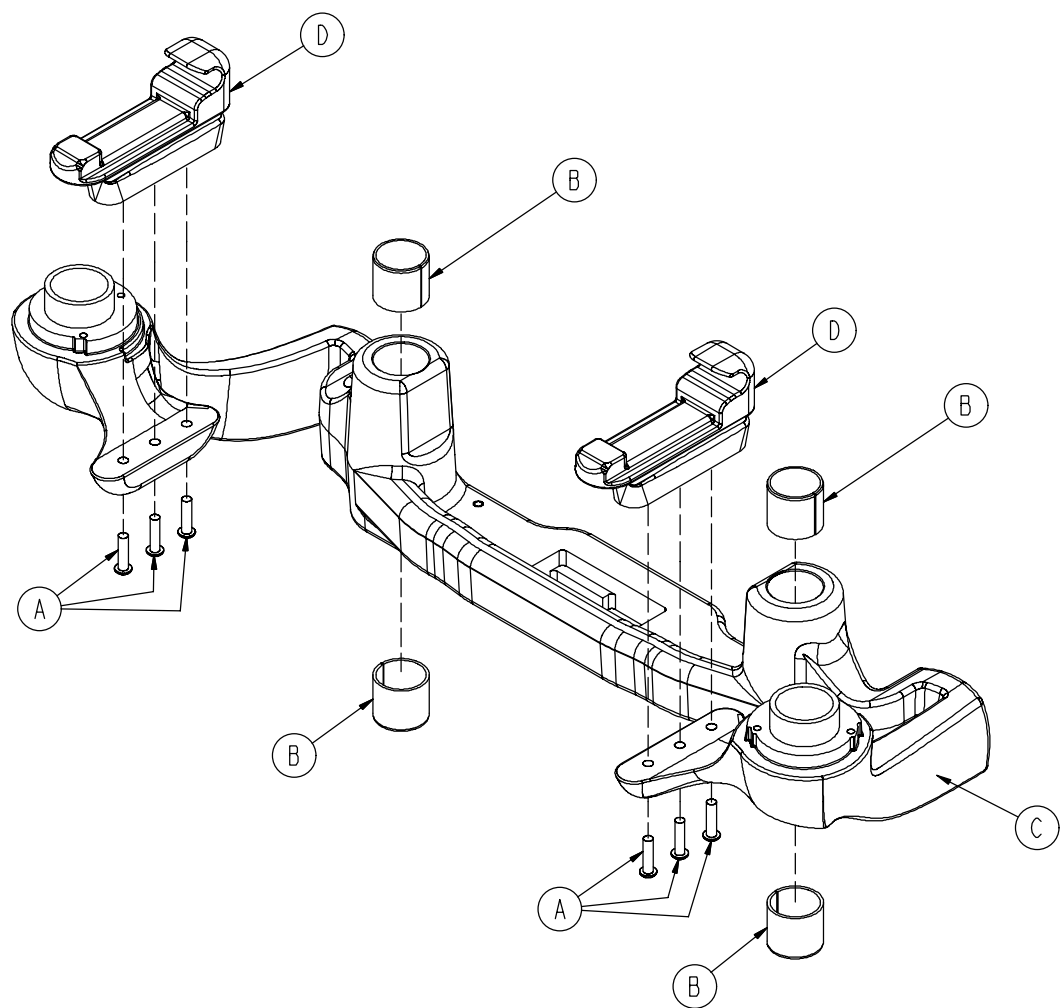


A	0004-440-000	6
B	0011-538-000	4
C	4701-040-002	1
D	4701-040-108	2

Rev B

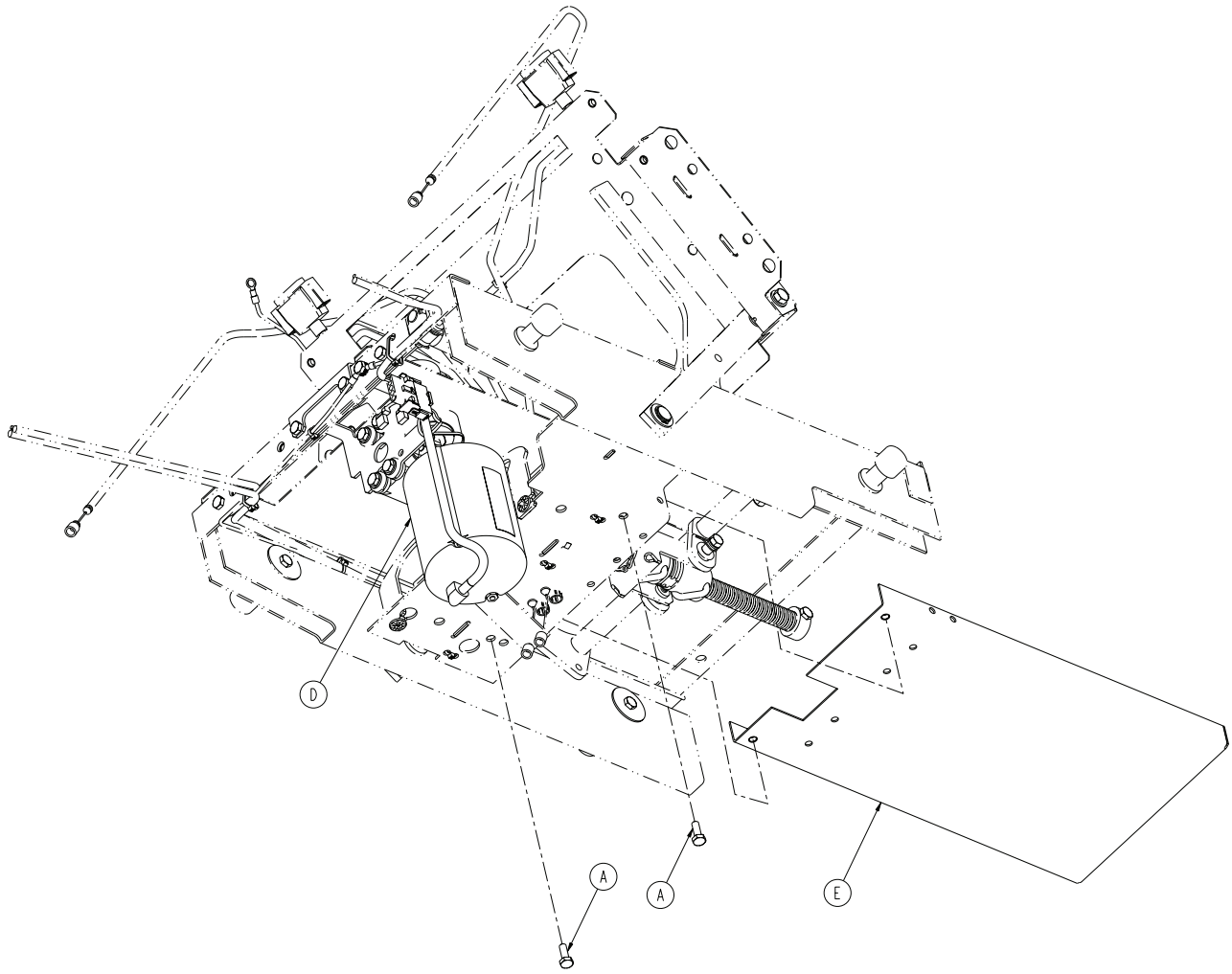


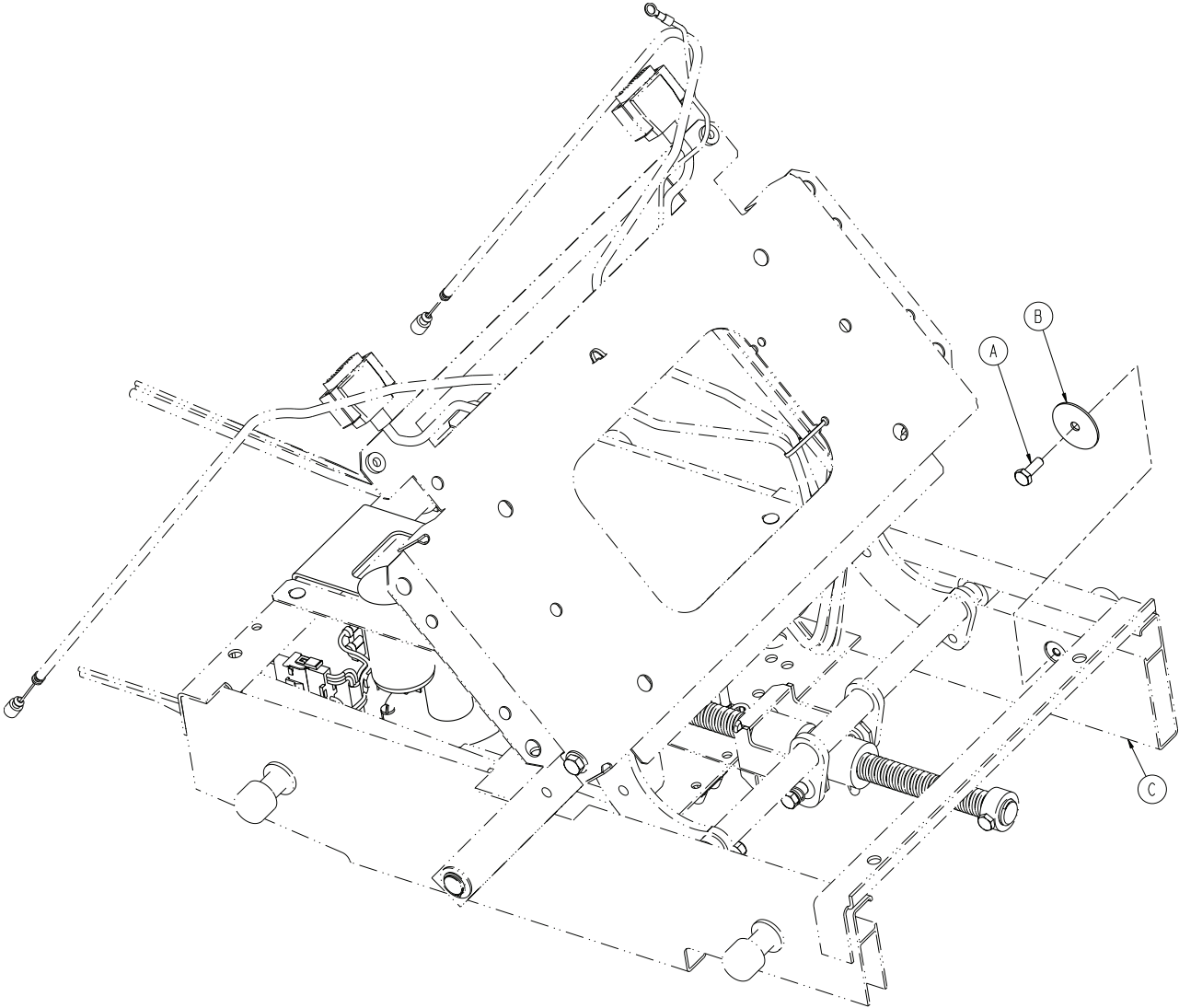


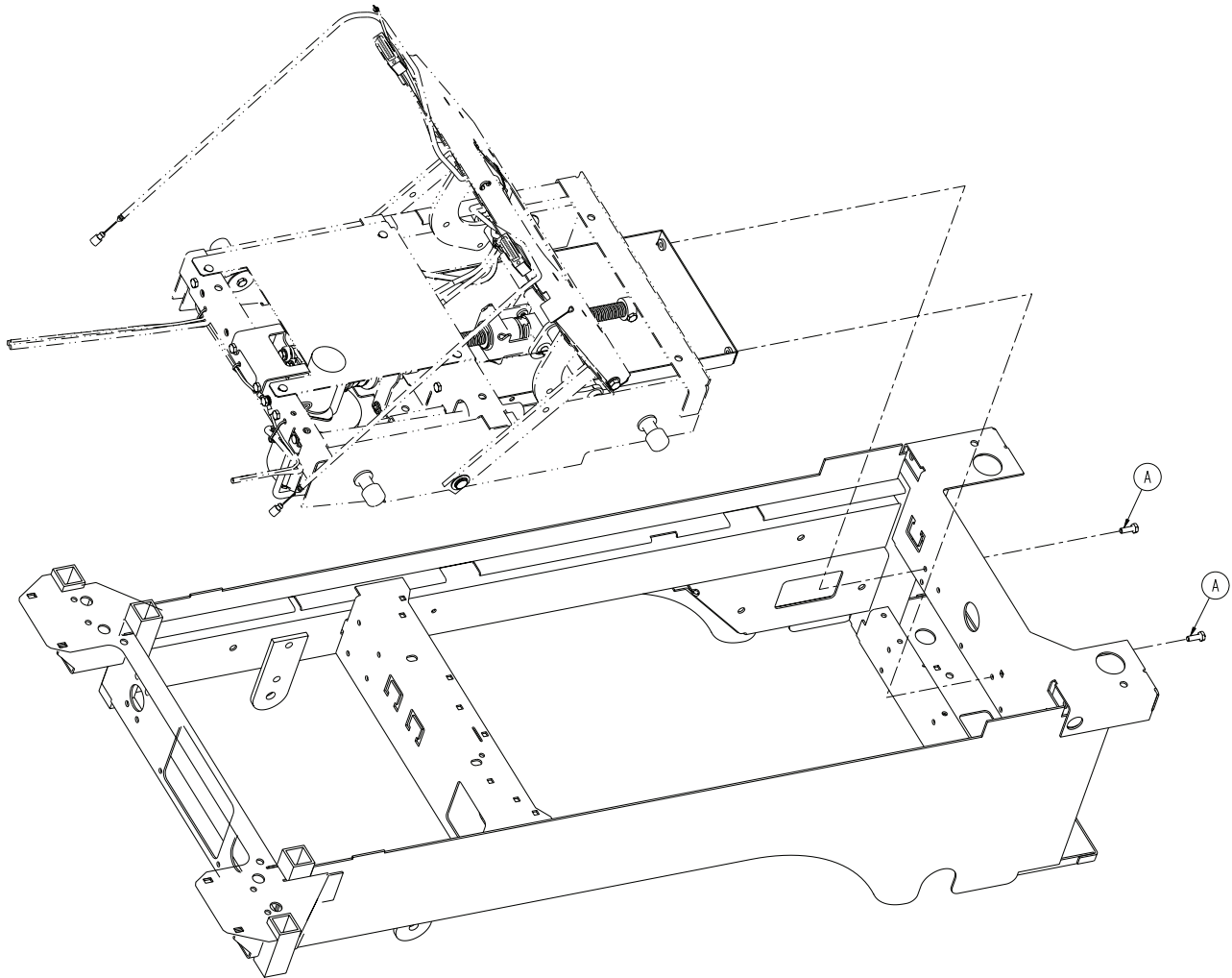


A	0003-214-000	5
B	0011-445-000	1
C	4701-035-000	1
D	4712-035-040	1
E	4701-035-046	1
F	4701-080-028	1

Rev F

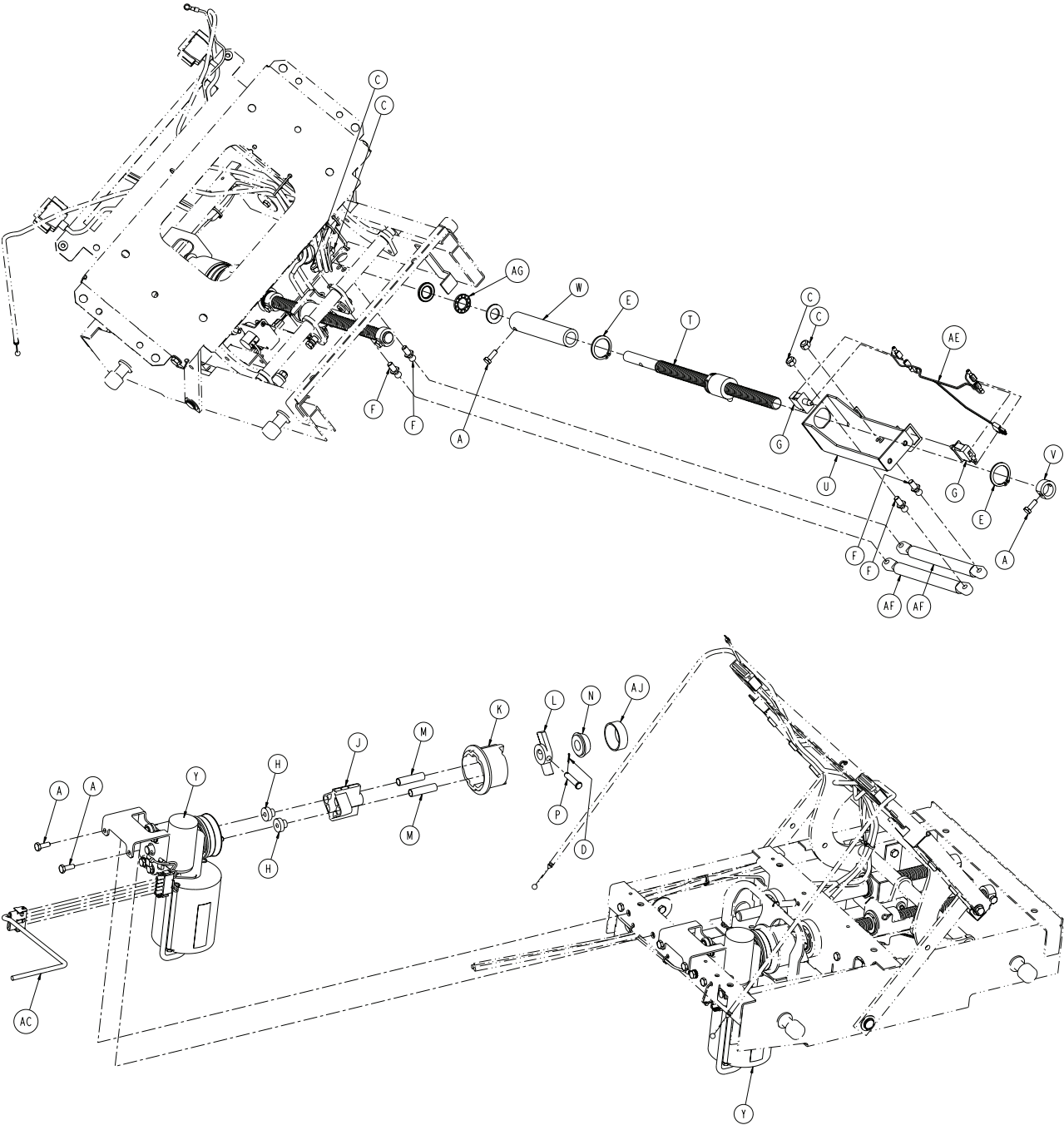

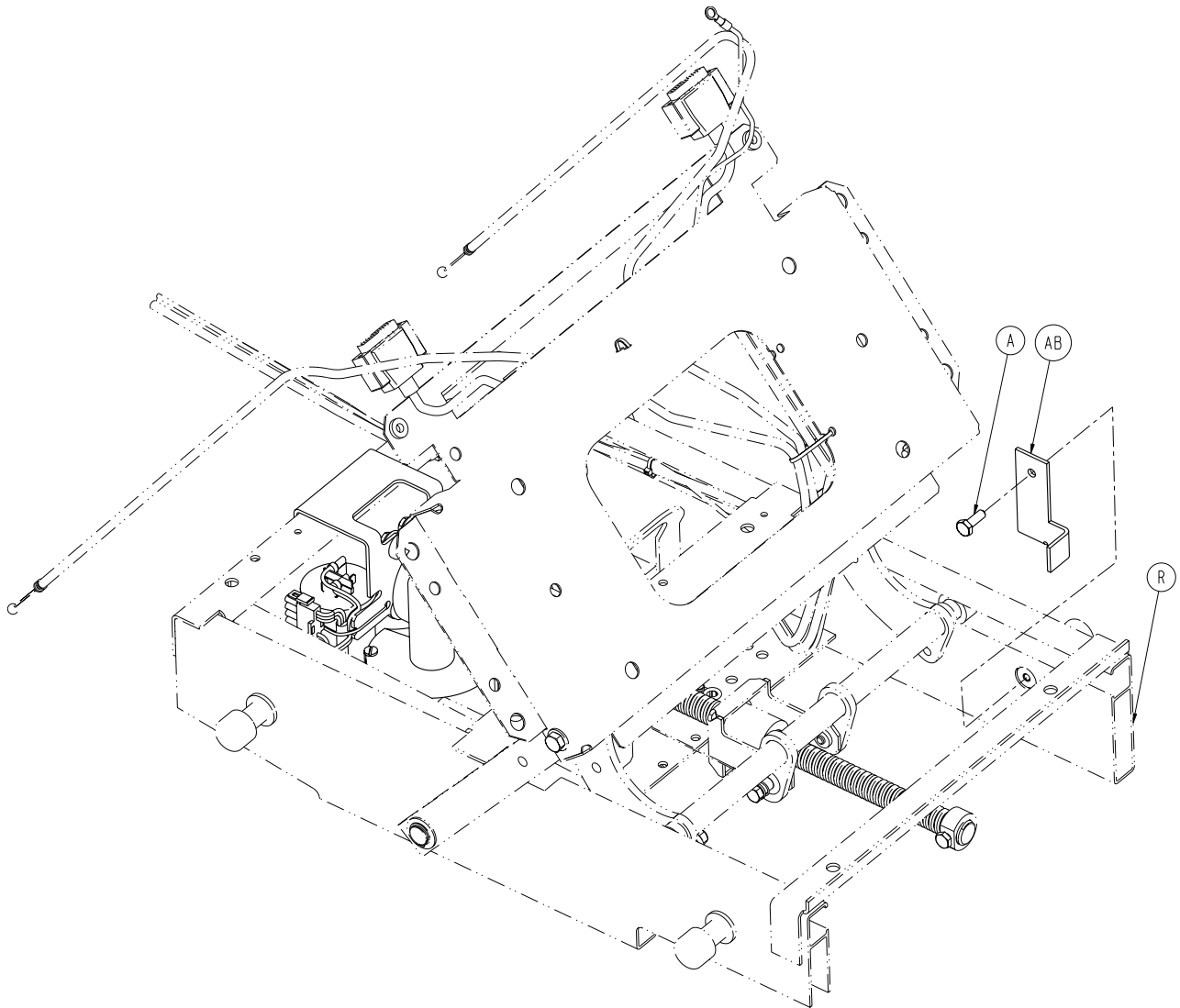


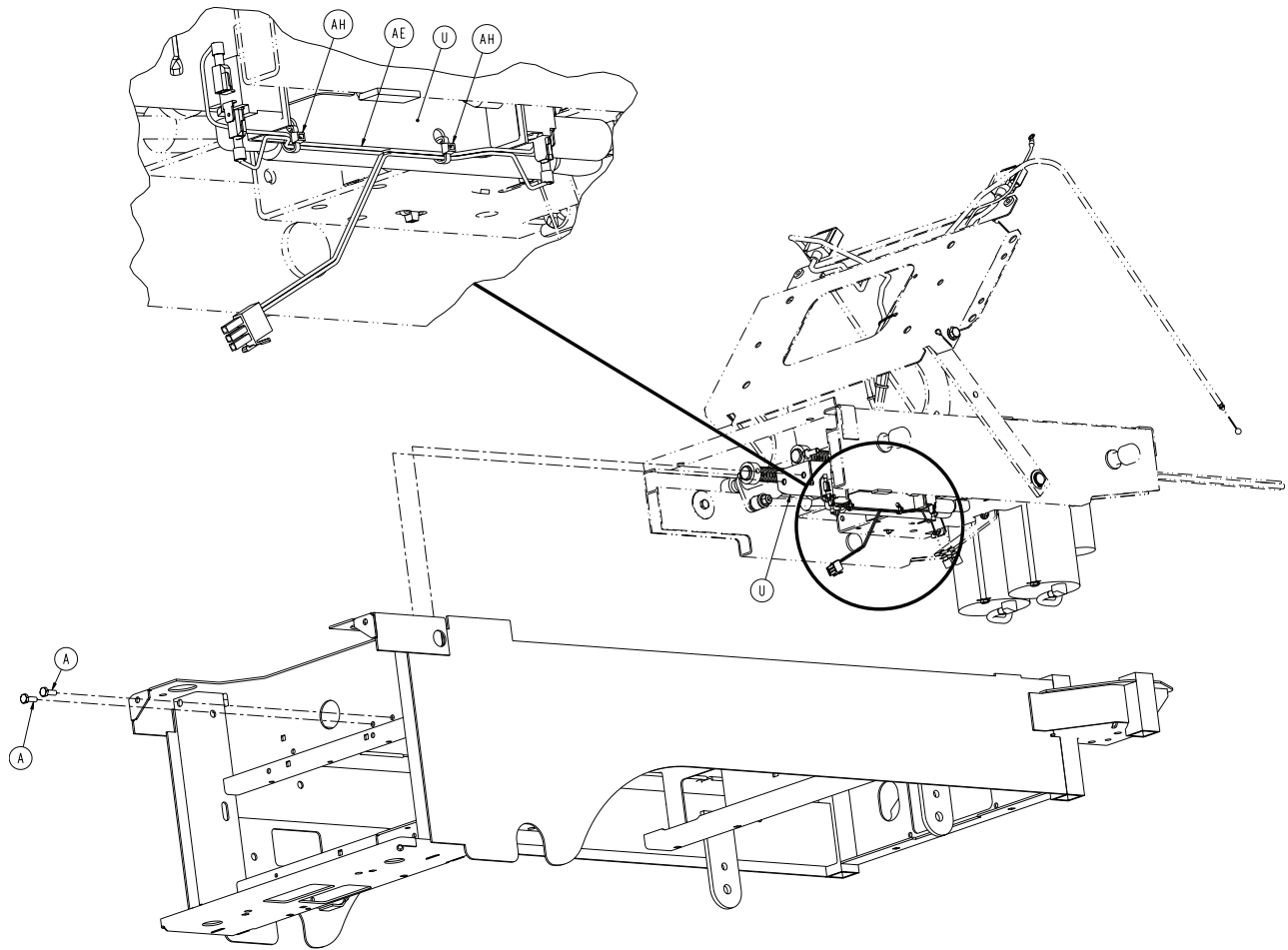



A	0003-214-000	7
C	0016-036-000	4
D	0027-017-000	1
E	0028-120-000	2
F	1550-031-002	4
G	3000-300-058	2
H	3000-300-455	2
J	3000-300-456	1
K	3000-300-461	1
L	3000-300-462	1
M	3000-300-464	2
N	3000-300-444	1
P	3000-300-473	1
R	4701-035-000	1
T	4701-035-012	1
U	4701-035-030	1


4712-035-100


V	5000-030-255	1
W	4701-035-023	1
Y	4712-035-040	2
AB	4701-035-042	1
AC	4701-080-014	1
AE	4701-080-016	1
AF	5000-030-213	2
AG	4701-035-047	1
AH	0059-086-000	2
AJ	3000-300-471	1

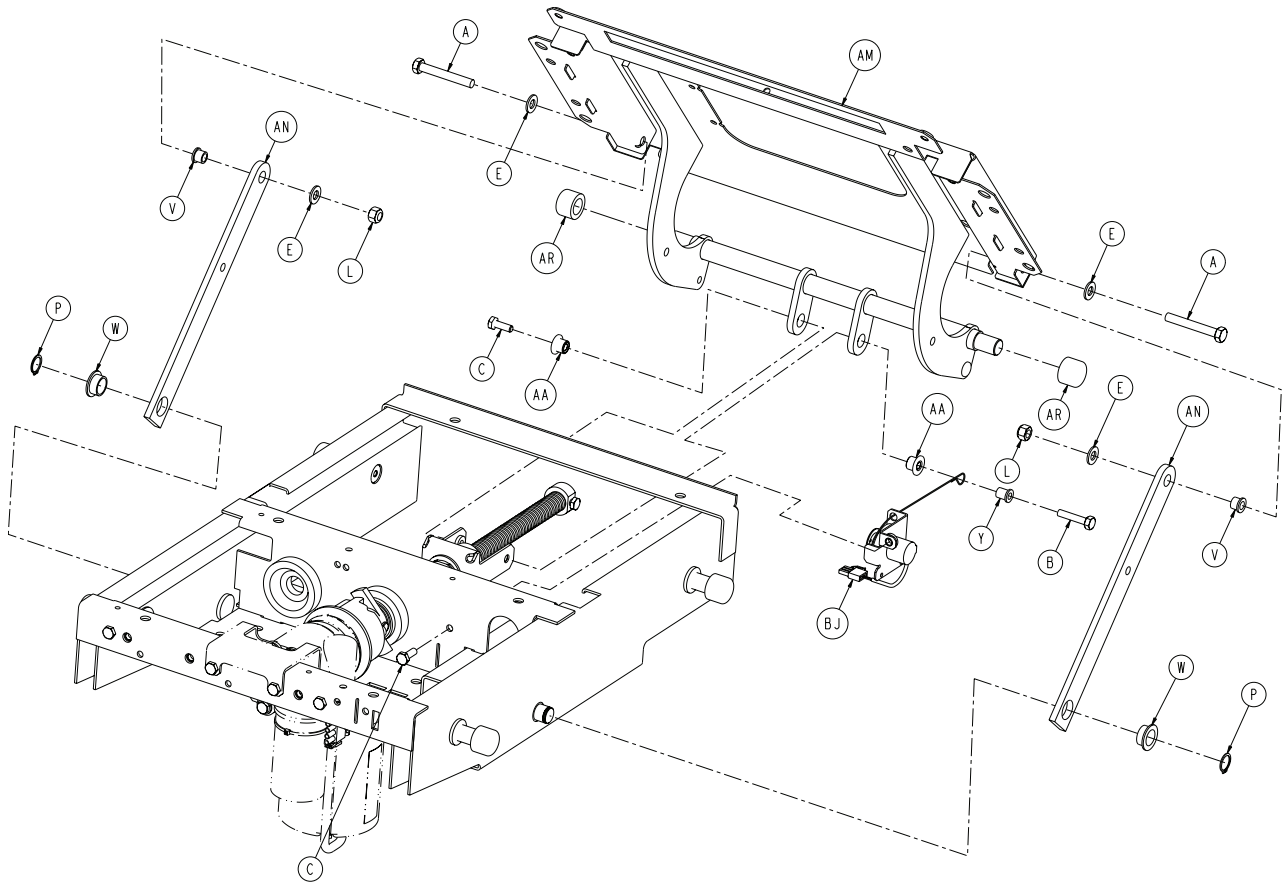


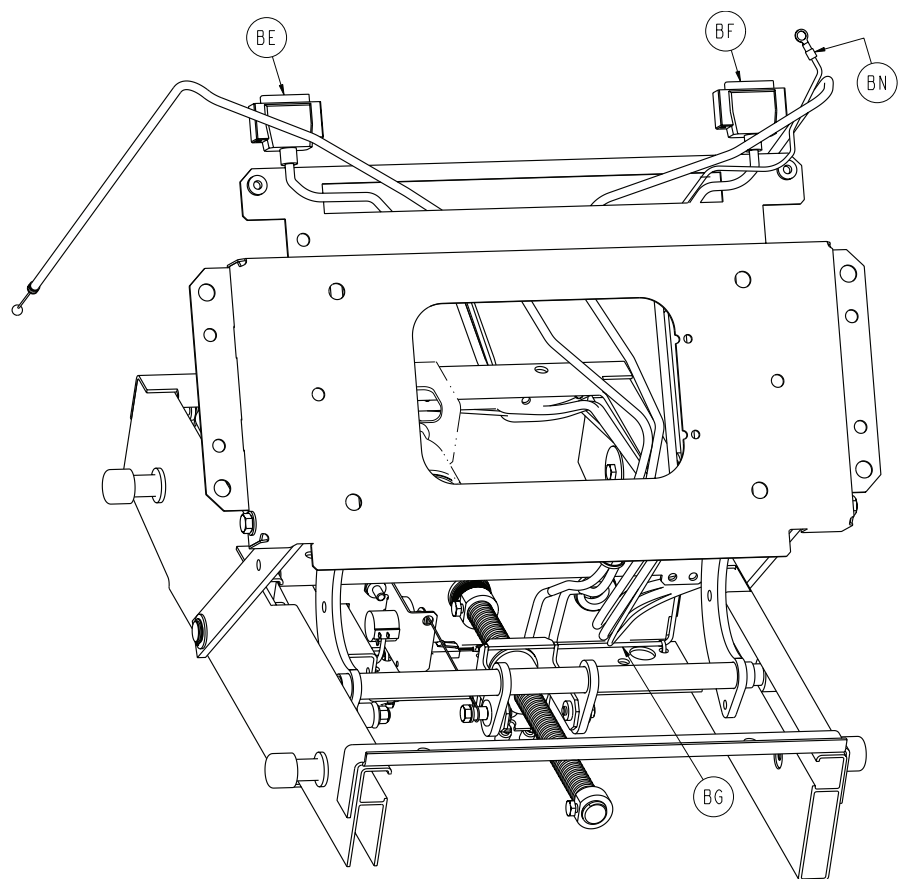


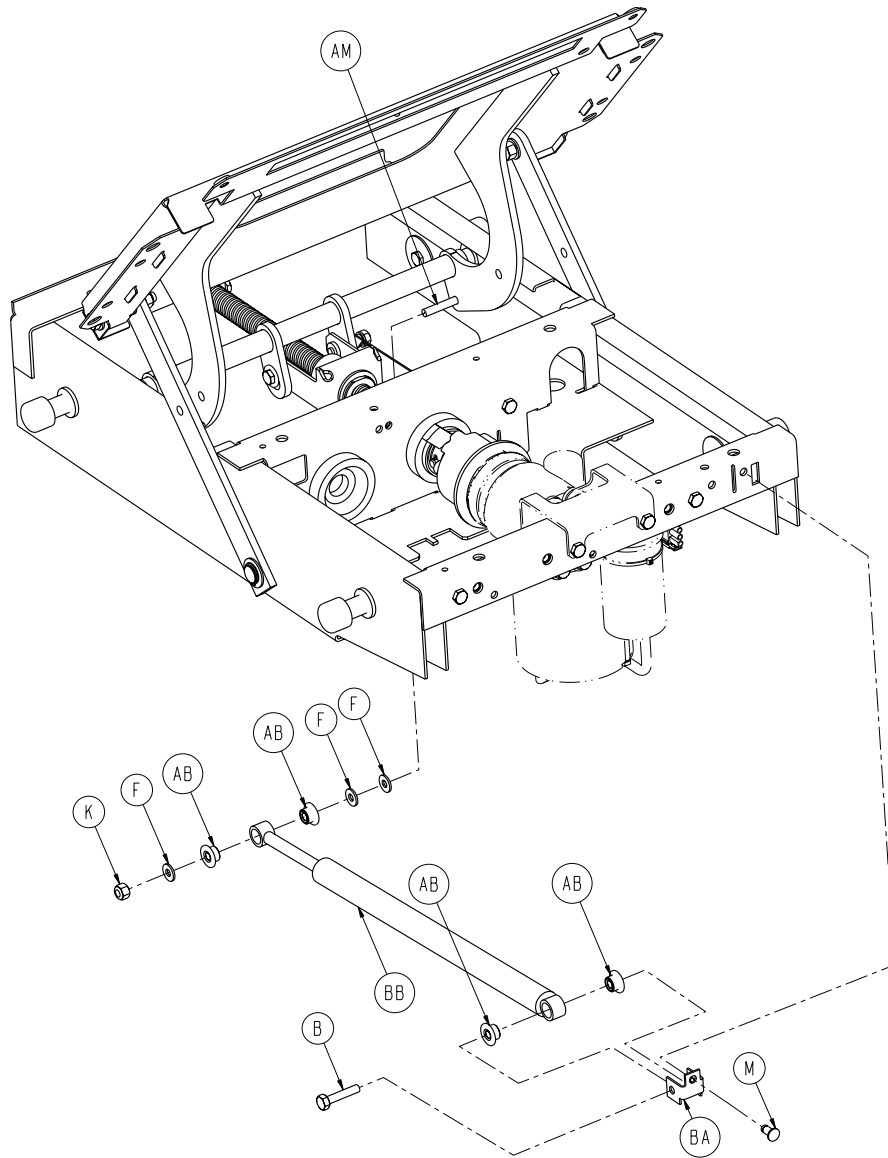


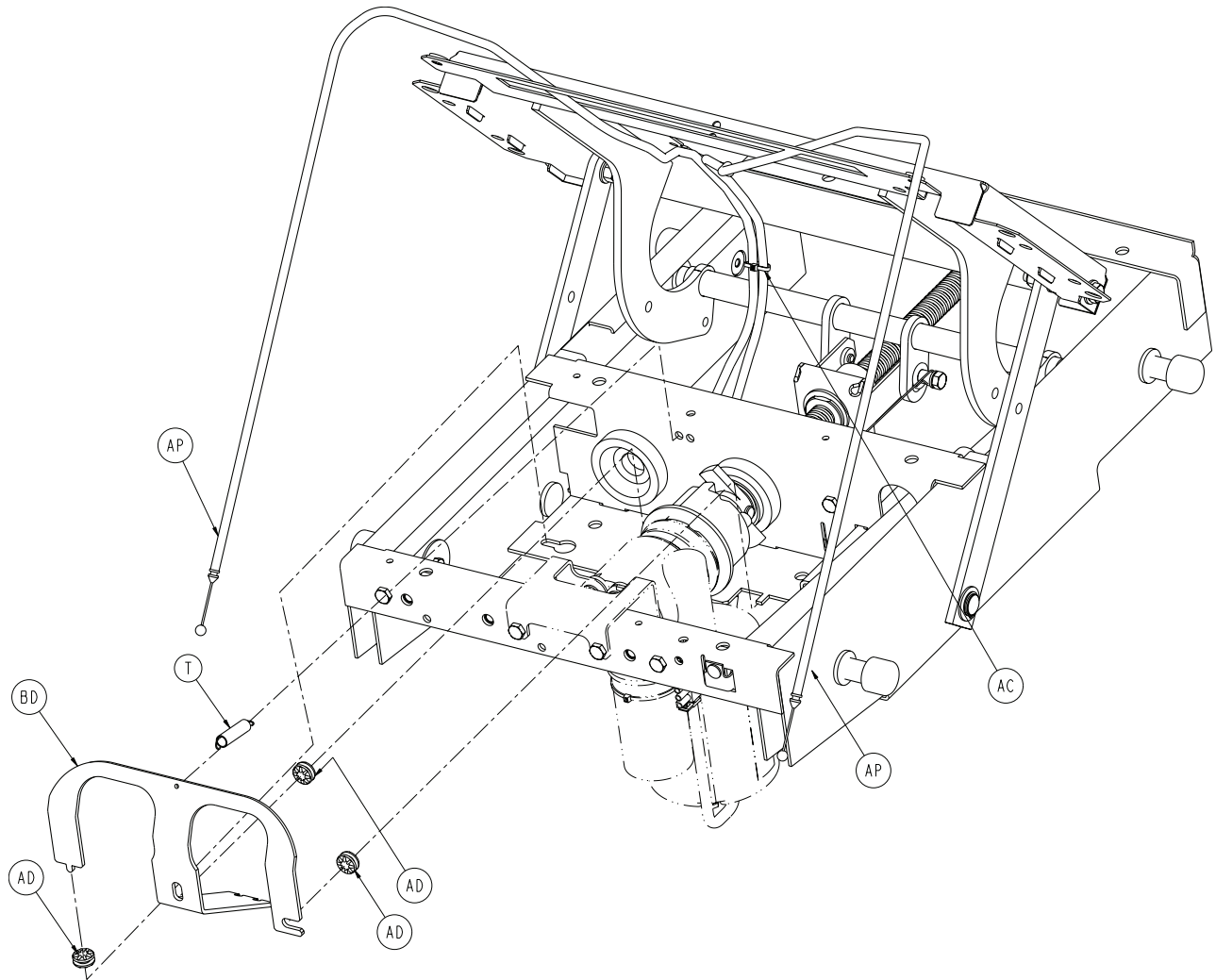


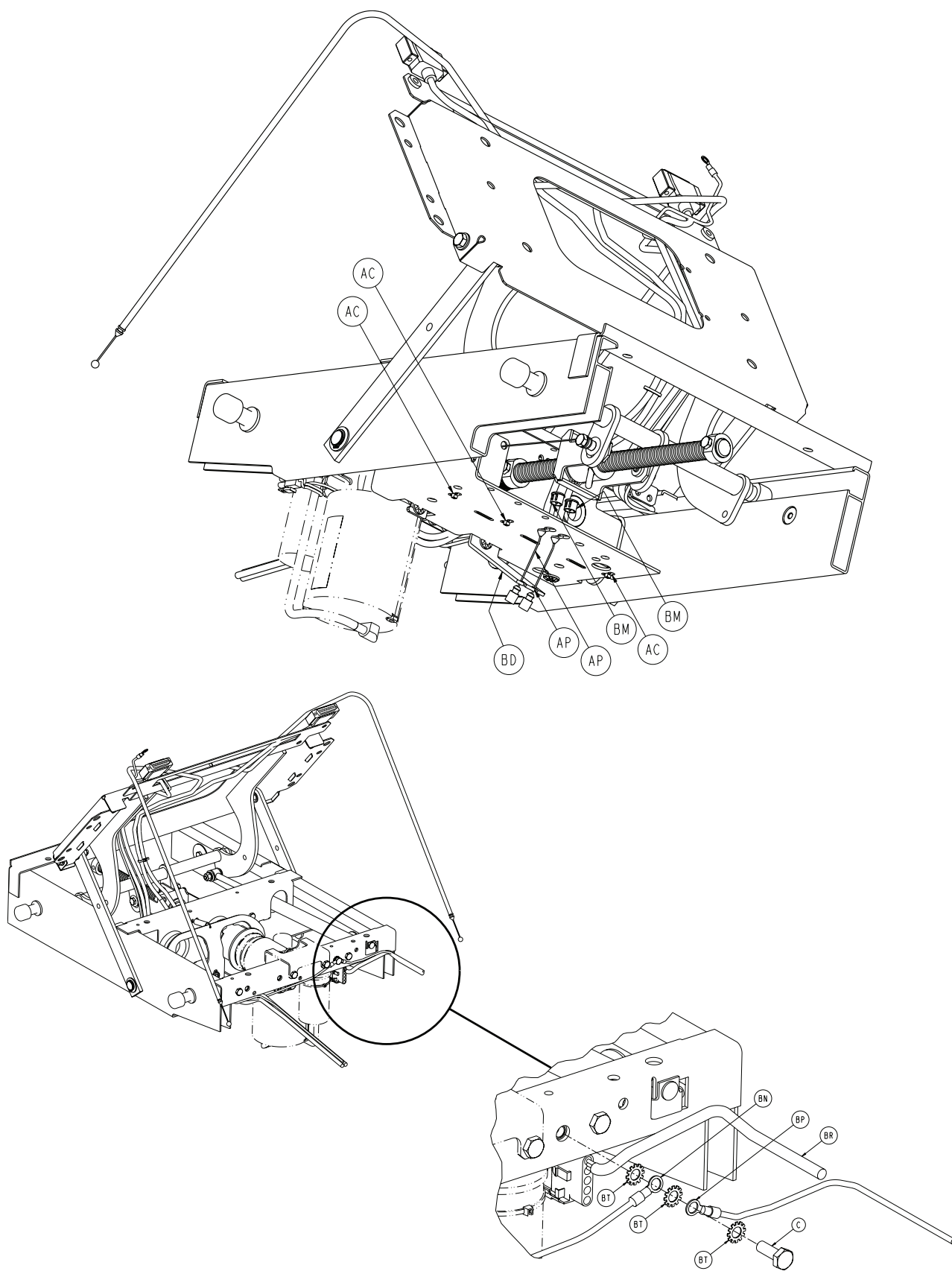

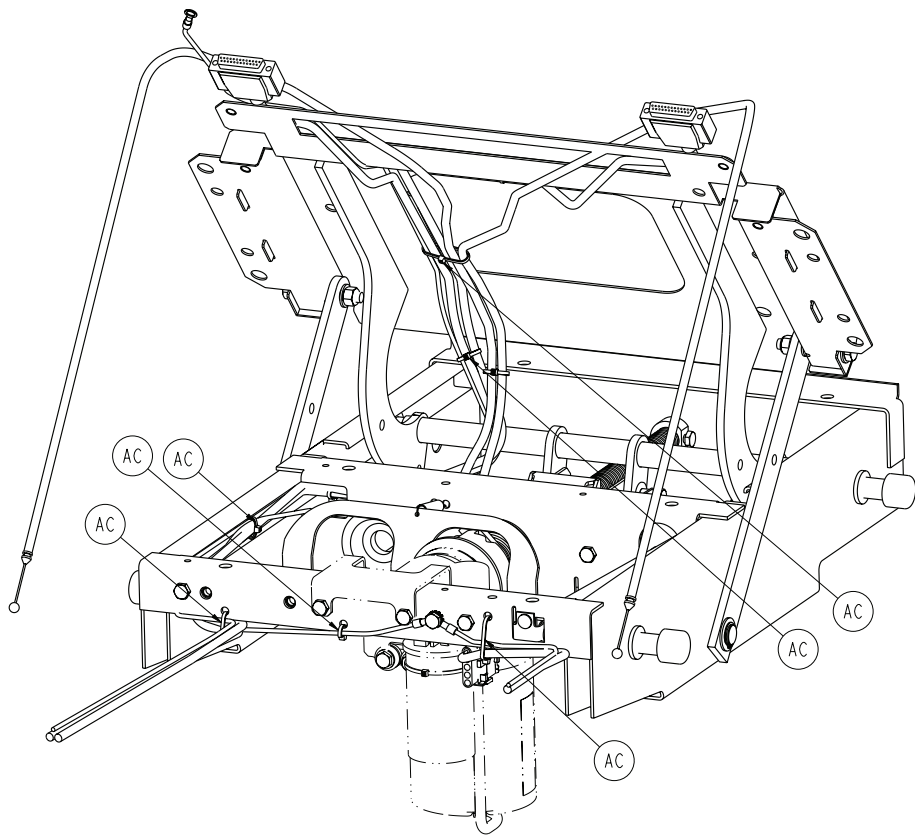

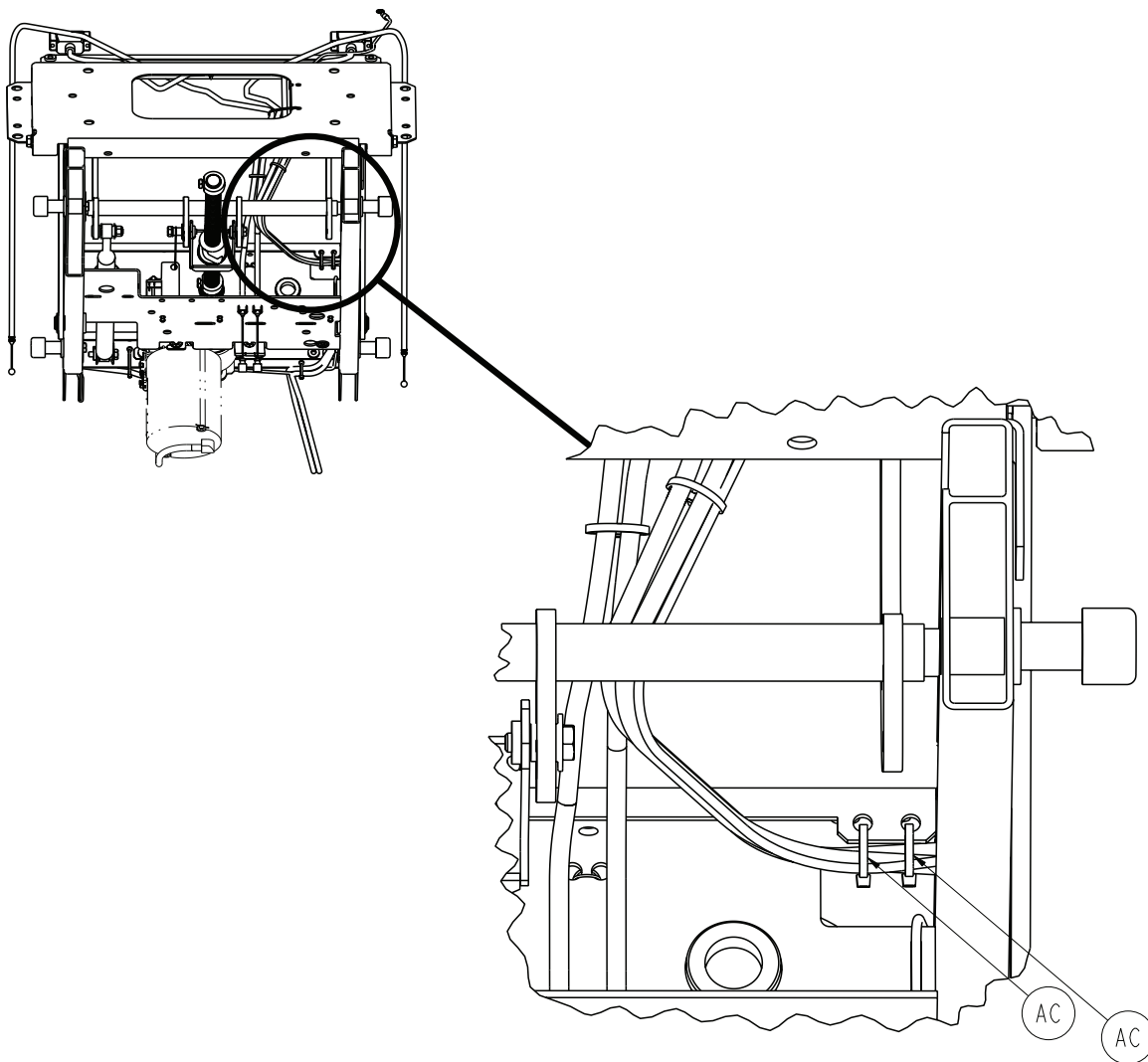

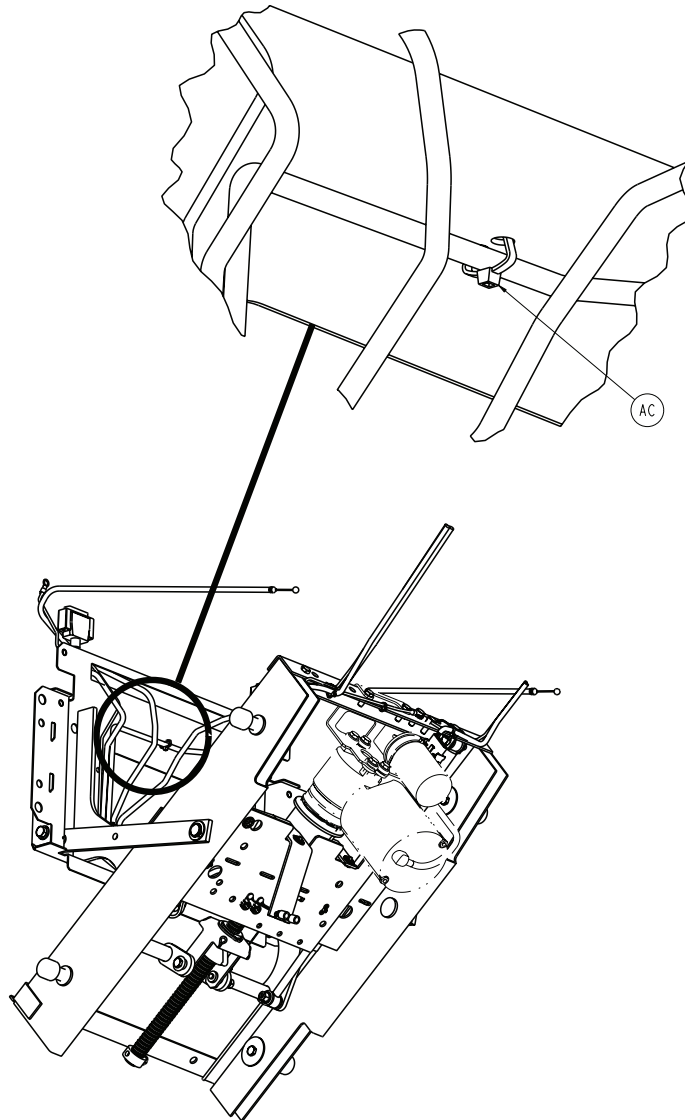

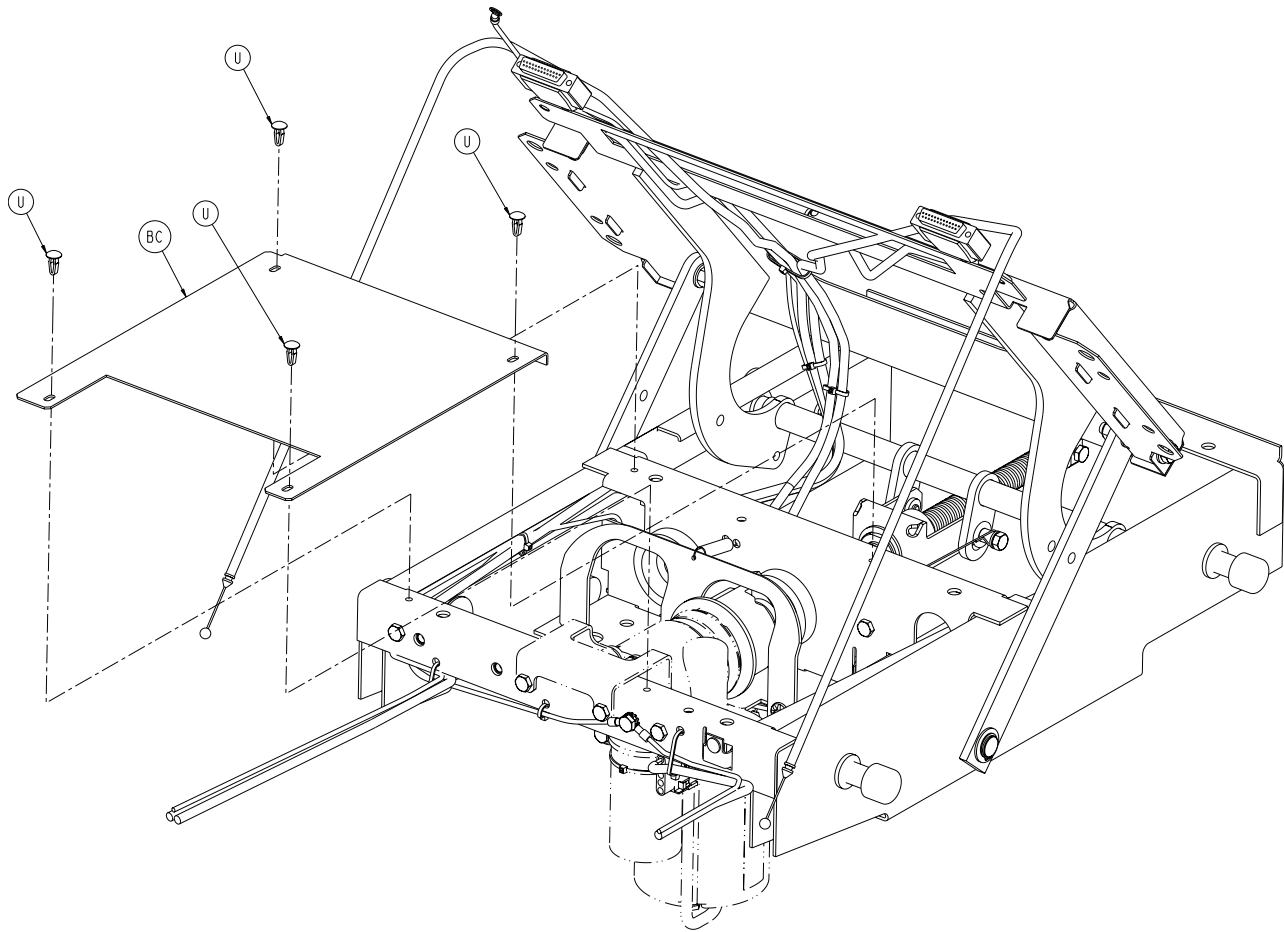



A	0003-062-000	2
B	0003-074-000	2
C	0003-214-000	12
E	0011-003-000	4
F	0011-063-000	3
H	0011-445-000	3
K	0016-028-000	1
L	0016-036-000	2
M	0025-120-000	1
N	0027-017-000	1
P	0028-023-000	2
R	0028-120-000	2
T	0038-404-000	1
U	0052-904-000	4
V	0081-222-000	2
W	0081-404-000	2
Y	0052-818-000	1

4701-035-000

AA	3000-300-001	2
AB	3000-300-099	4
AC	0059-086-000	13
AD	3000-300-442	3
AE	3000-300-455	2
AF	3000-300-456	1
AG	3000-300-461	1
AH	3000-300-462	1
AJ	3000-300-464	2
AK	3000-300-444	1
AL	3000-300-473	1
AM	4701-030-120	1
AN	4701-030-122	2
AP	4701-030-163	2
AR	4701-035-003	6
AT	4701-035-005	4
AU	4701-035-010	1
AV	4701-035-012	1
AW	4701-035-016	1
AY	4701-035-017	1
BA	4701-035-020	1
BB	4701-035-027	1
BC	4701-035-044	1
BD	4701-035-161	1
BE	4701-080-001	1
BF	4701-080-002	1
BG	4701-080-013	1
BJ	4701-080-025	1
BK	5000-030-255	2
BL	3000-300-466	1
BM	0030-052-000	2
BN	4701-080-027	1
BP	4701-080-018	1
BR	4701-080-015	1
BT	0013-010-000	3
BU	0014-083-000	5
BV	3000-300-471	1


Rev D

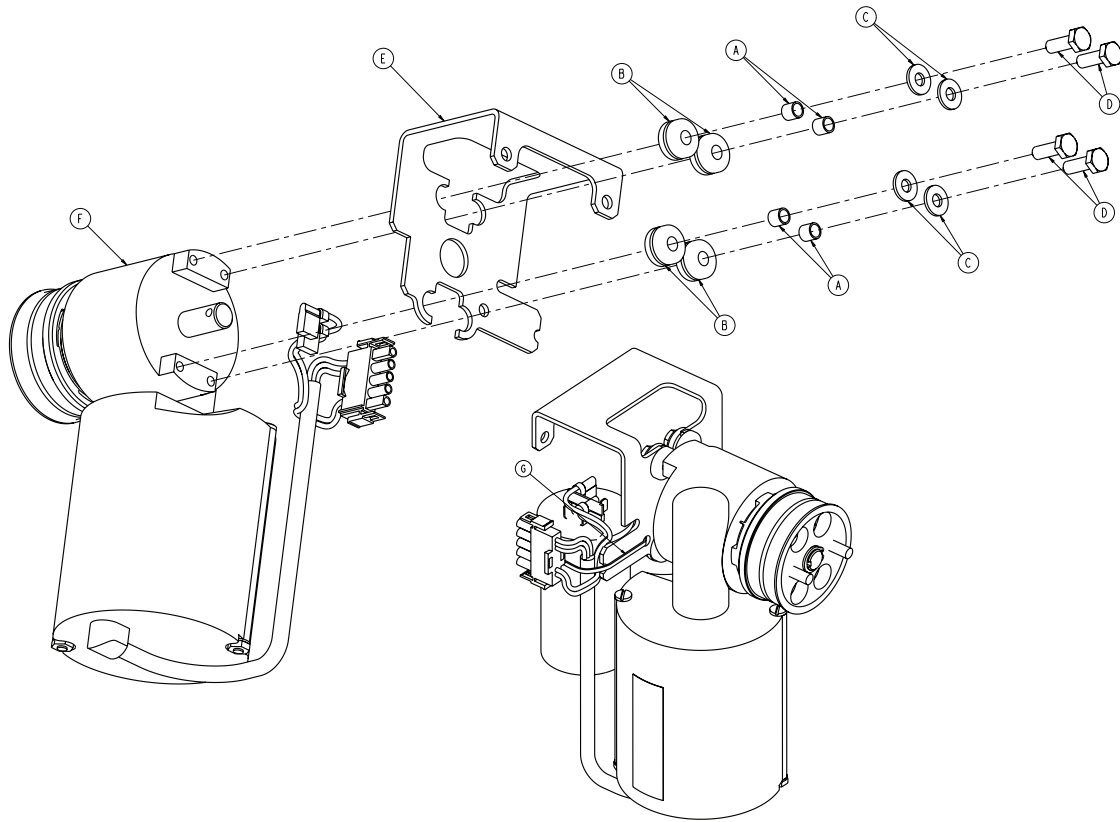

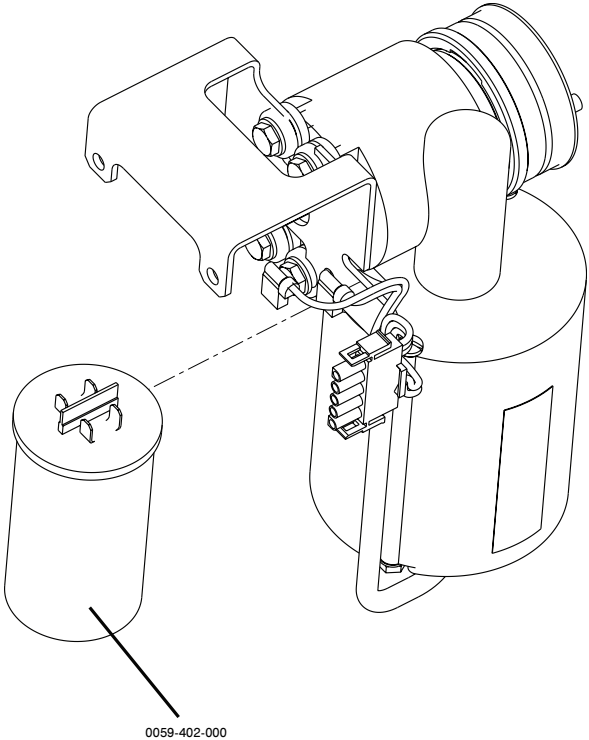

A	0715-001-333	4
B	0030-053-000	4
C	0011-063-000	4
D	0003-214-000	4
E	4701-035-036	1
F	4712-035-045	1
G	0059-086-000	1


Rev C

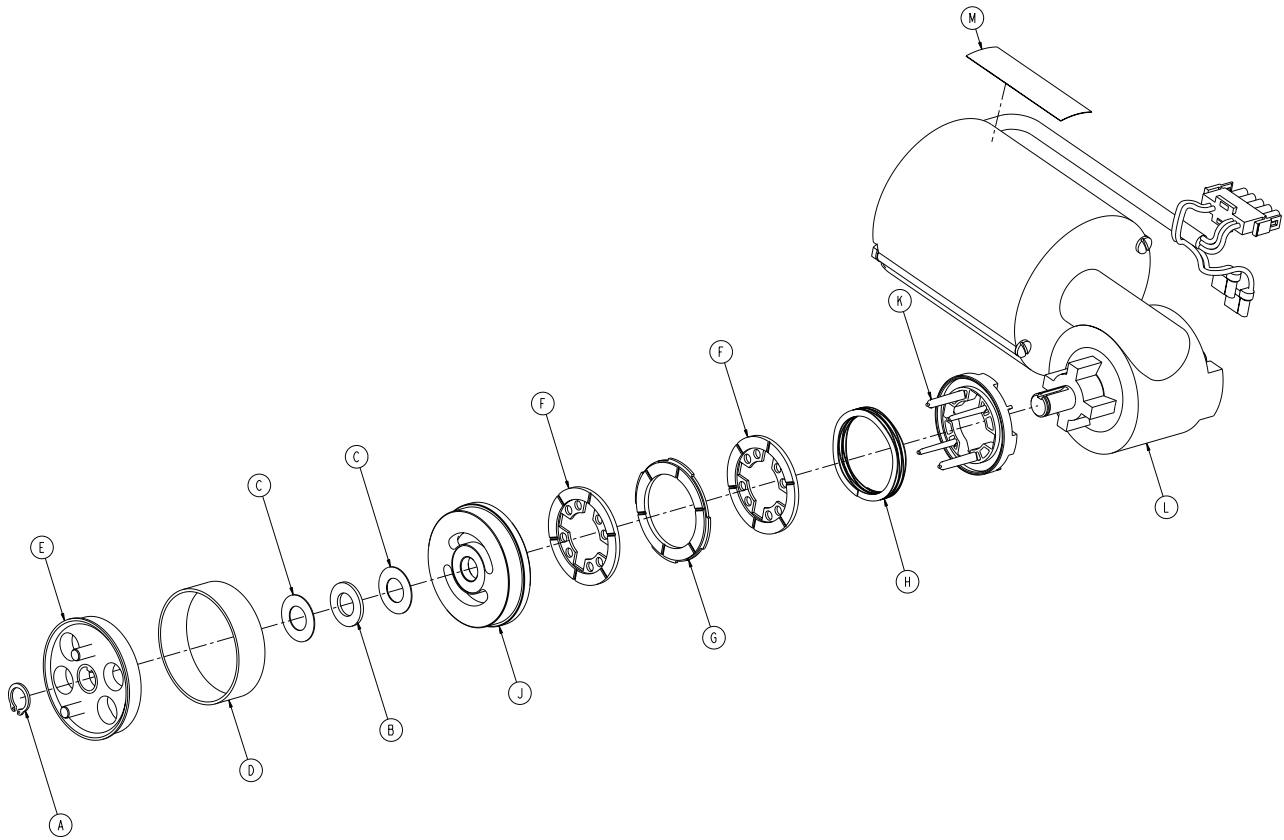

A	0028-131-000	1
B	0081-212-000	1
C	3000-200-224	2
D	3000-300-465	1
E	3001-300-455	1
F	3001-300-551	2
G	3001-300-552	1
H	3001-300-563	1
J	3001-300-569	1
K	3001-300-570	1
L	4712-116-004	1
M	4701-090-056	1


4701-140-240/4701-140-245


Rev A / Rev A

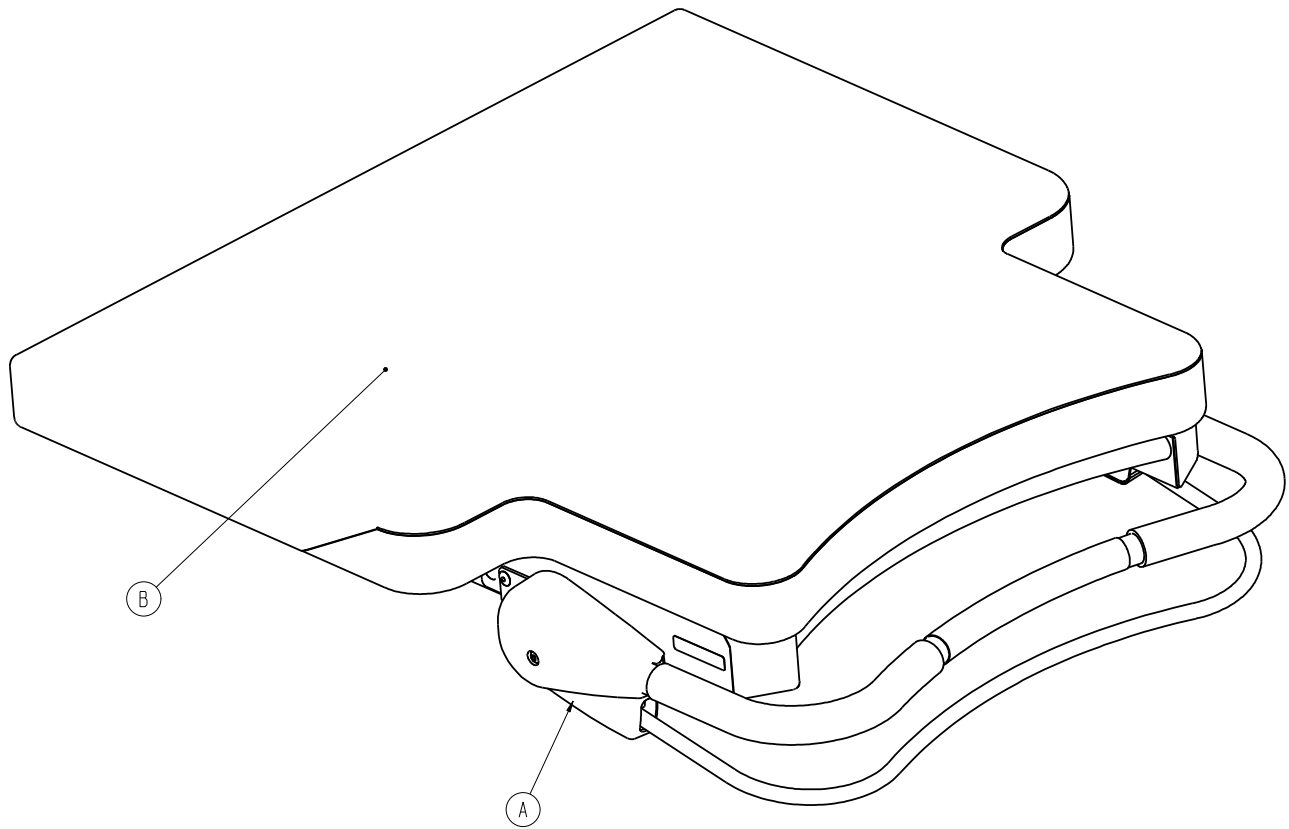



A	4701-140-005	1
B	4701-045-005	1
	4701-045-015	1


Rev A

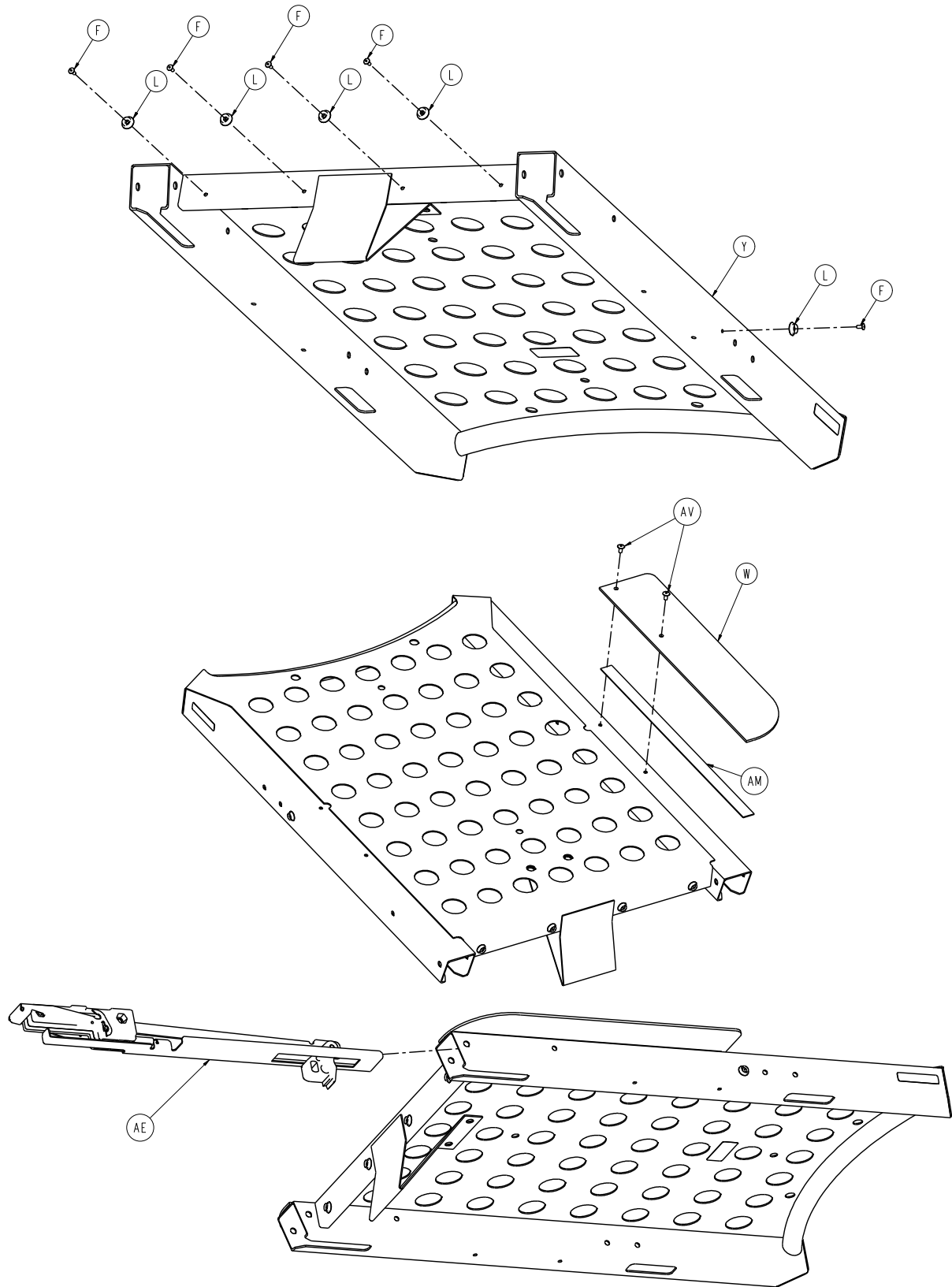

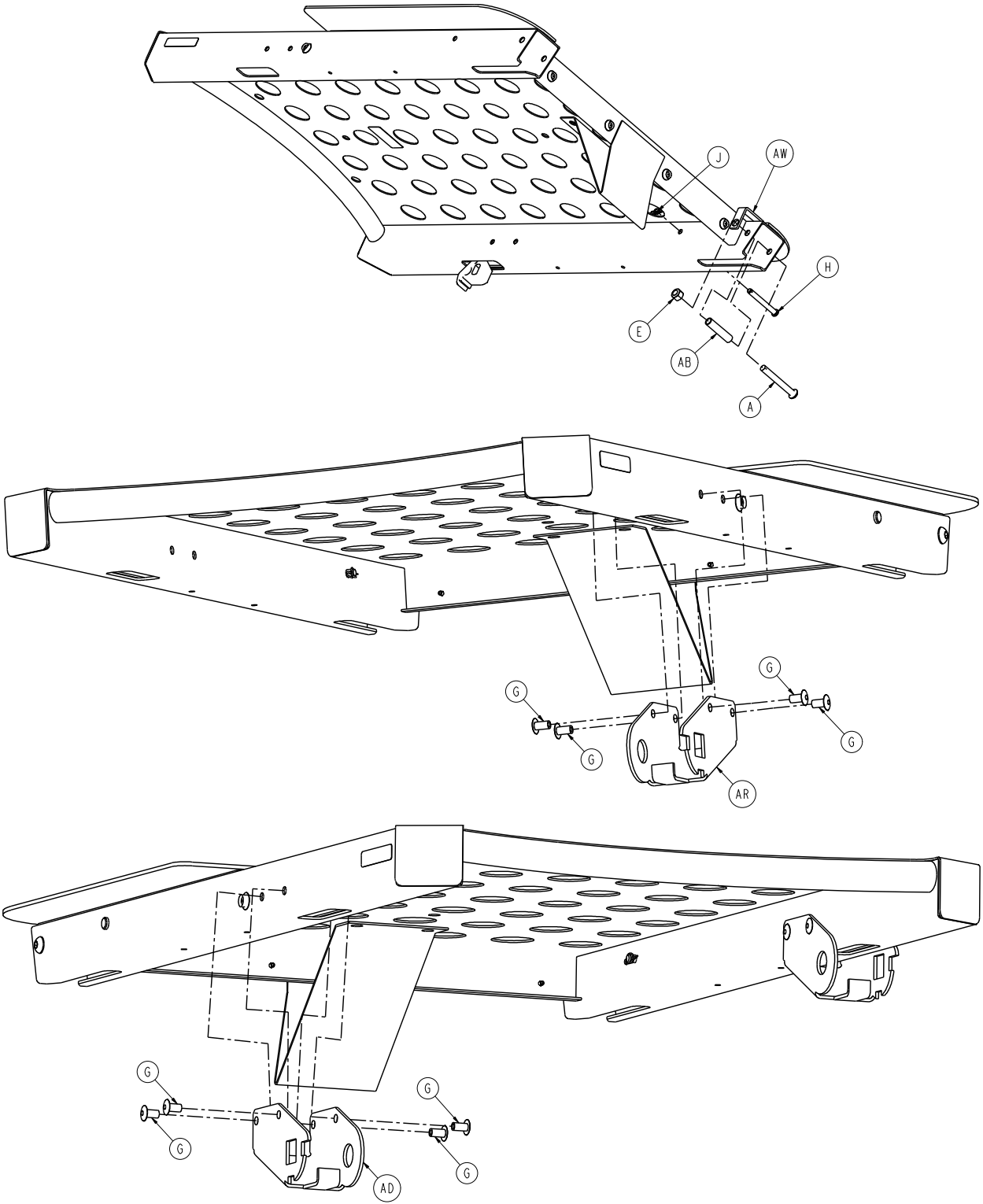

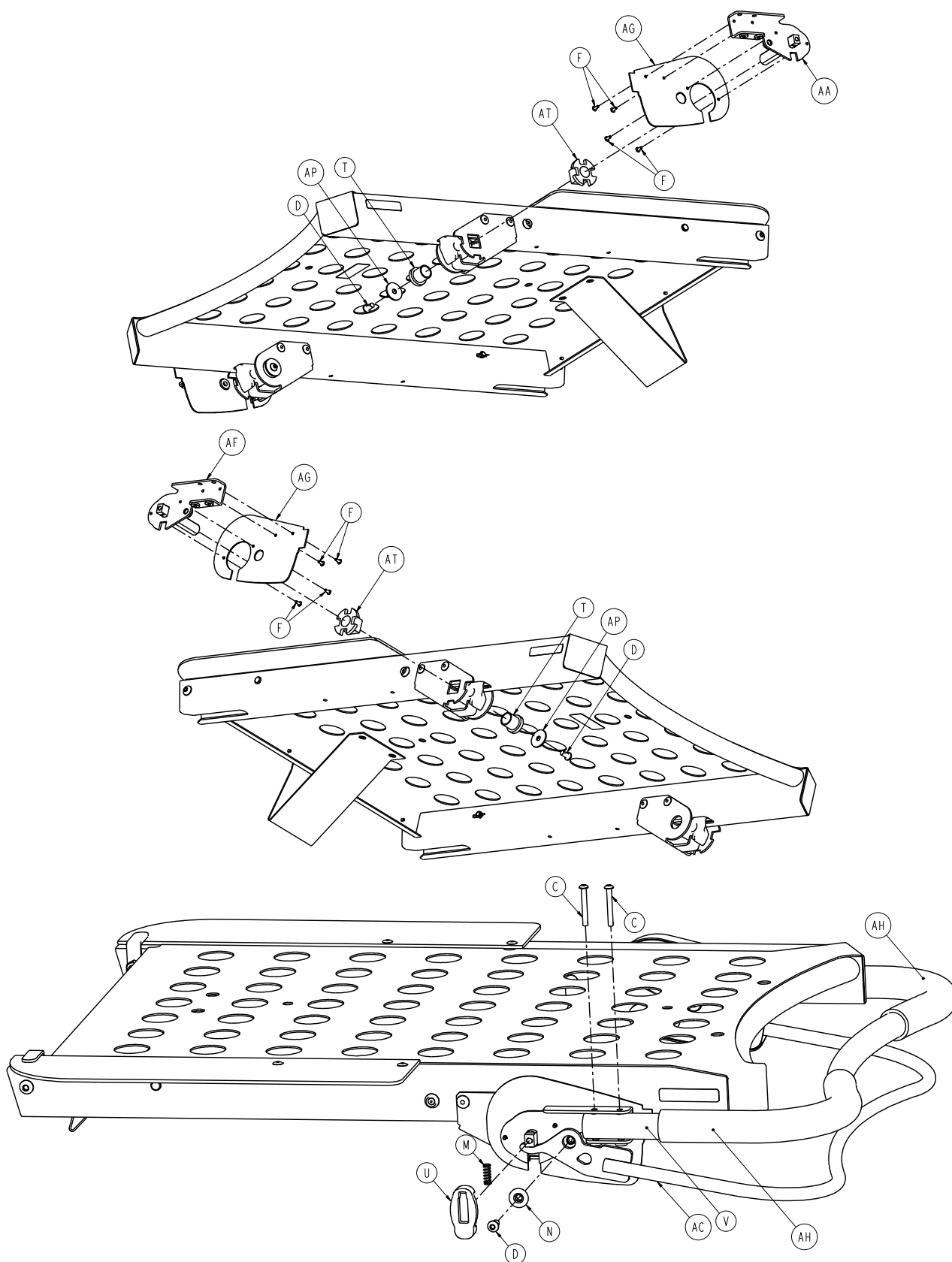

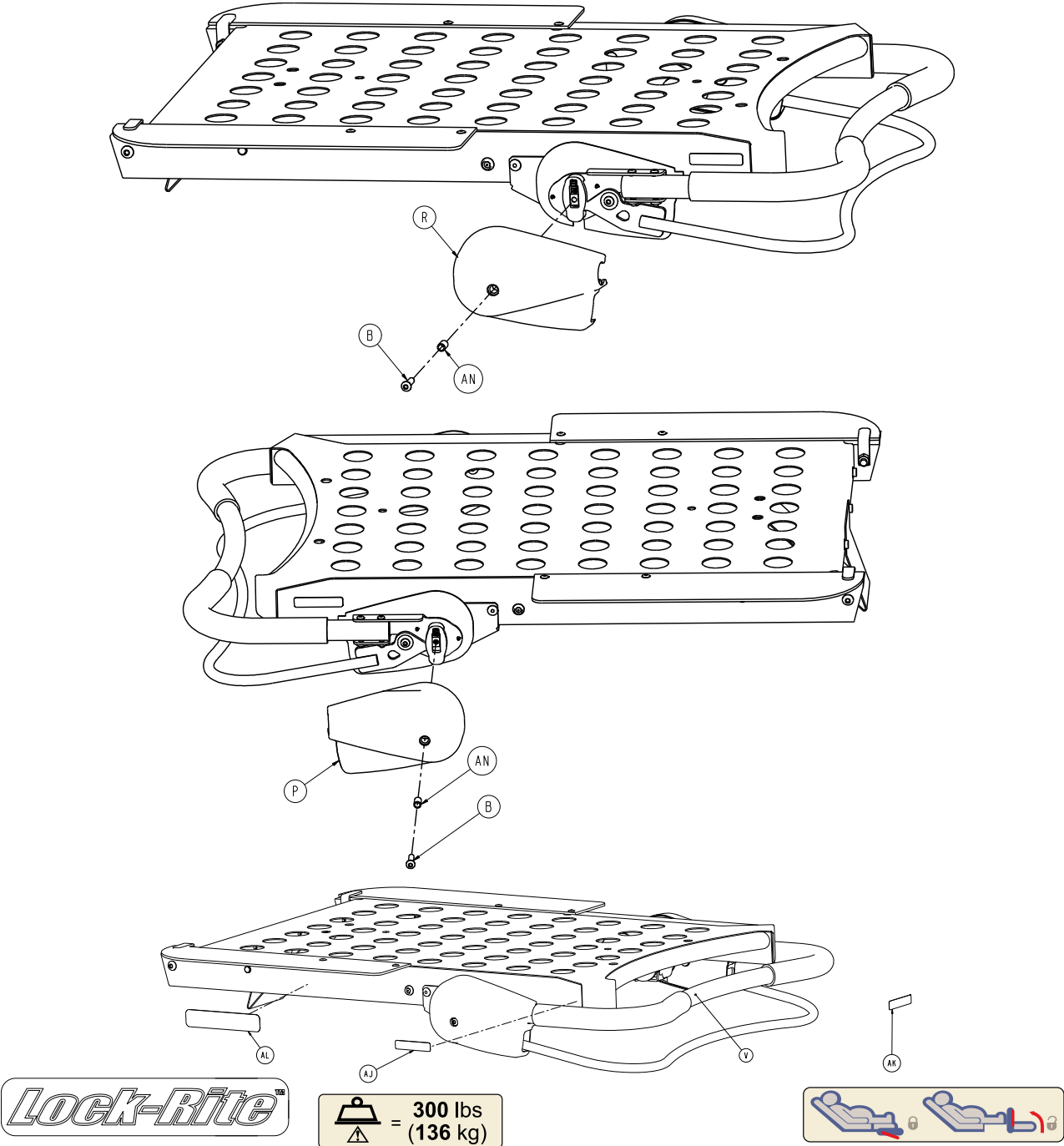

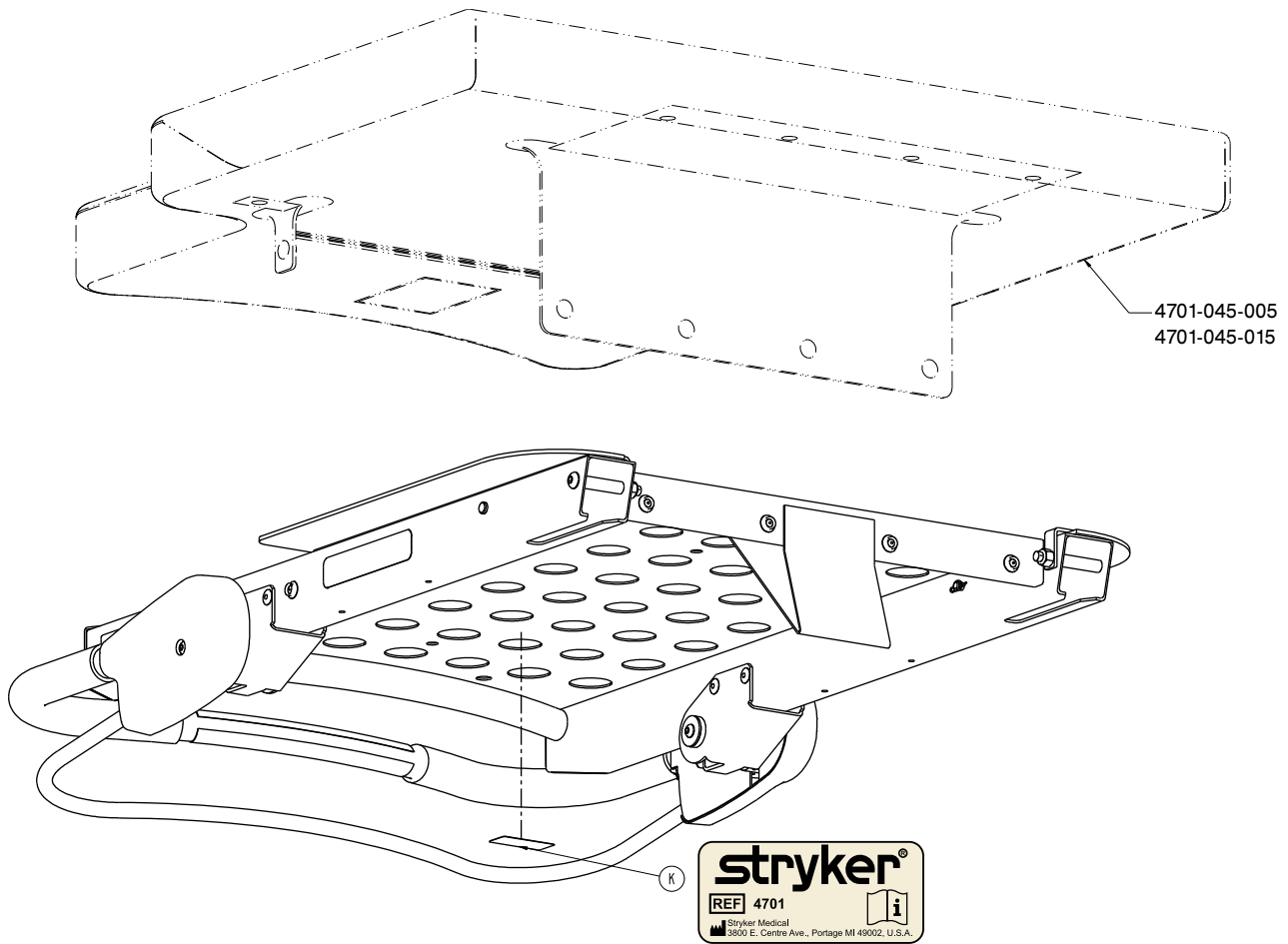


A	0004-512-000	2
B	0004-334-000	2
C	0004-314-000	4
D	0004-325-000	2
E	0016-036-000	2
F	0025-038-000	14
G	0025-177-000	8
H	0026-334-000	2
J	0027-020-000	2
K	4701-109-981	1
L	0029-003-000	6
M	0038-220-000	2
N	0081-268-000	2
P	4701-040-003	1
R	4701-040-004	1
T	4701-040-006	2



4701-140-005


U	4701-040-007	2
V	4701-040-008	1
W	4701-040-009	1
Y	4701-040-010	1
AA	4701-040-015	1
AB	4701-040-019	2
AC	4701-040-020	1
AD	4701-040-023	1
AE	4701-040-025	2
AF	4701-040-040	1
AG	4701-040-109	2
AH	4701-040-129	2
AJ	4701-109-967	2
AK	4701-109-944	1
AL	4701-090-043	2
AM	0044-054-000	2
AN	4701-040-162	2
AP	0011-053-000	2
AR	4701-040-164	1
AT	4701-040-166	2
AU	4701-040-085	1
AV	0025-079-000	4
AW	4701-040-179	2




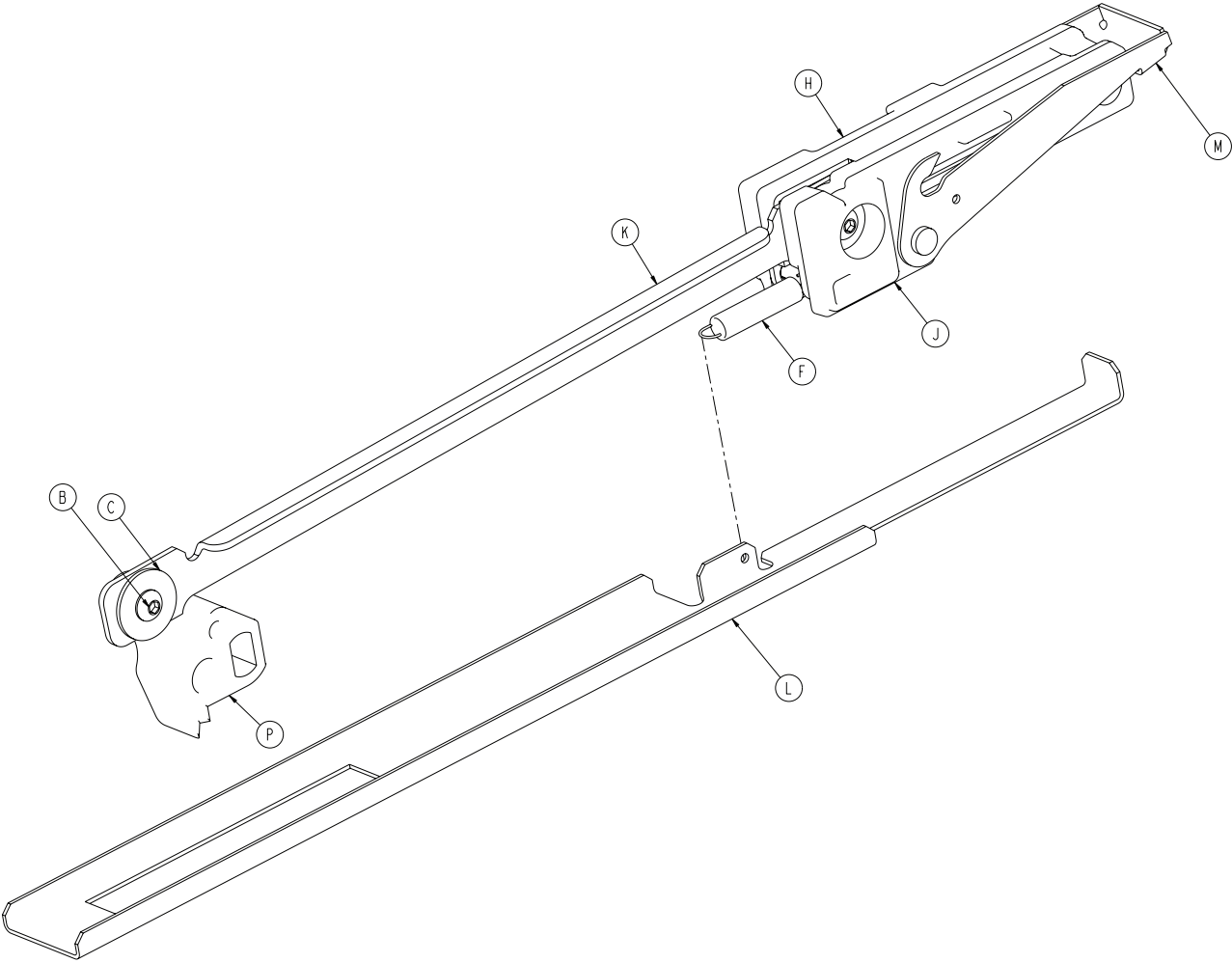


A	0004-334-000	2
B	0004-325-000	1
C	0011-053-000	1
D	0016-028-000	2
E	0027-020-000	1
F	0038-427-000	1
G	0081-268-000	1
H	4701-040-026	1
J	4701-040-027	1
K	4701-040-028	1
L	4701-040-029	1
M	4701-040-031	1
N	4701-040-032	1
P	4701-040-033	1

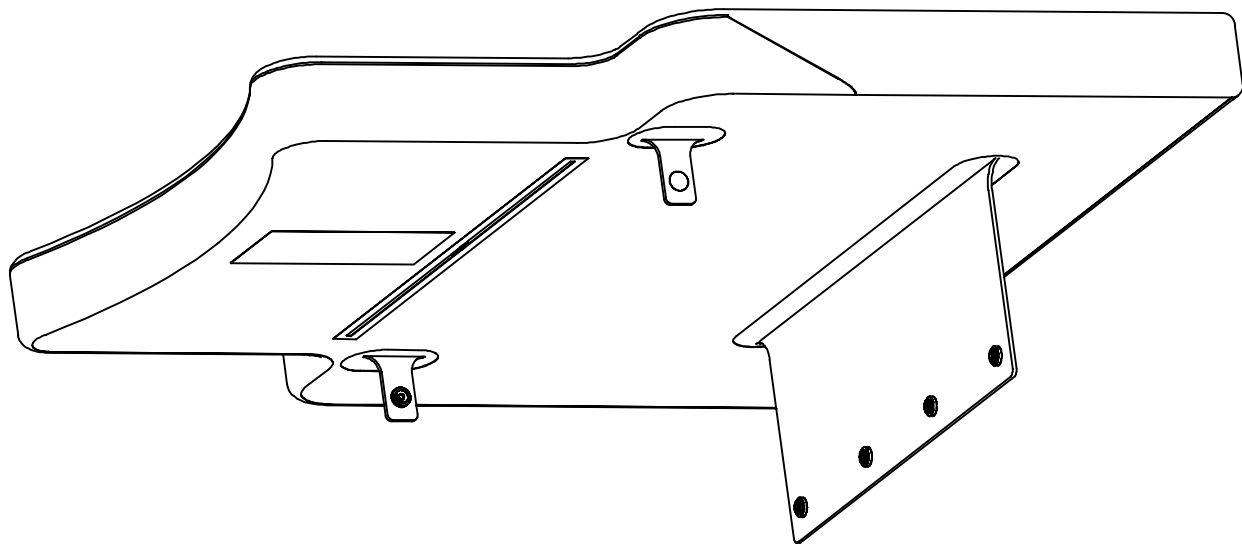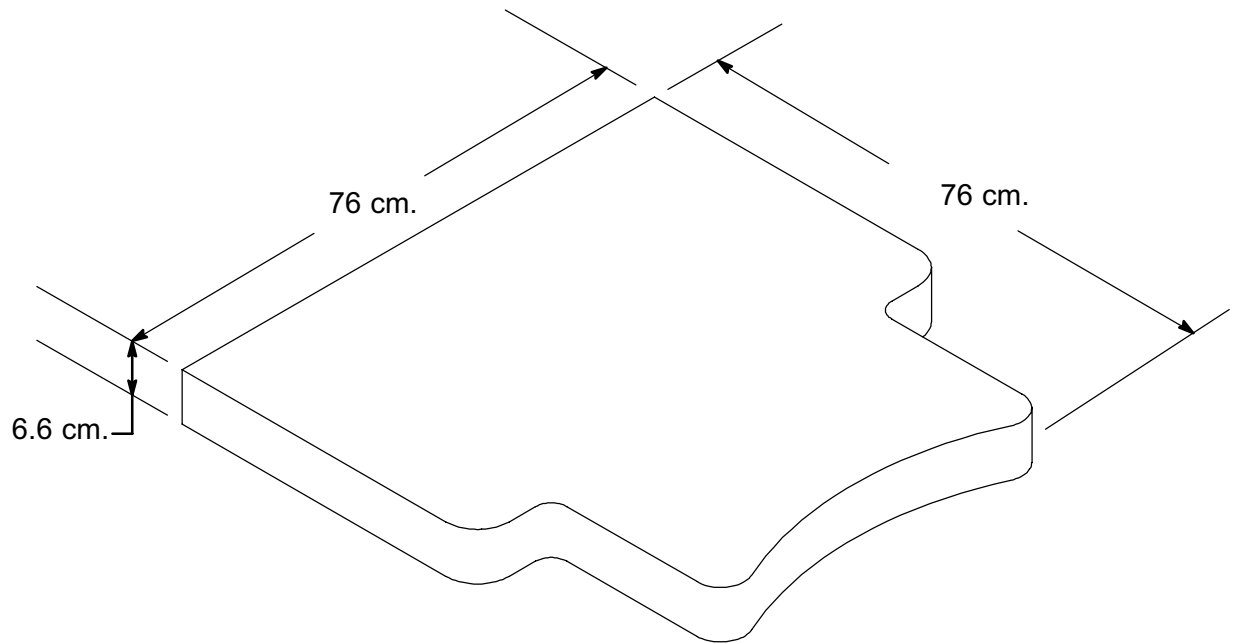


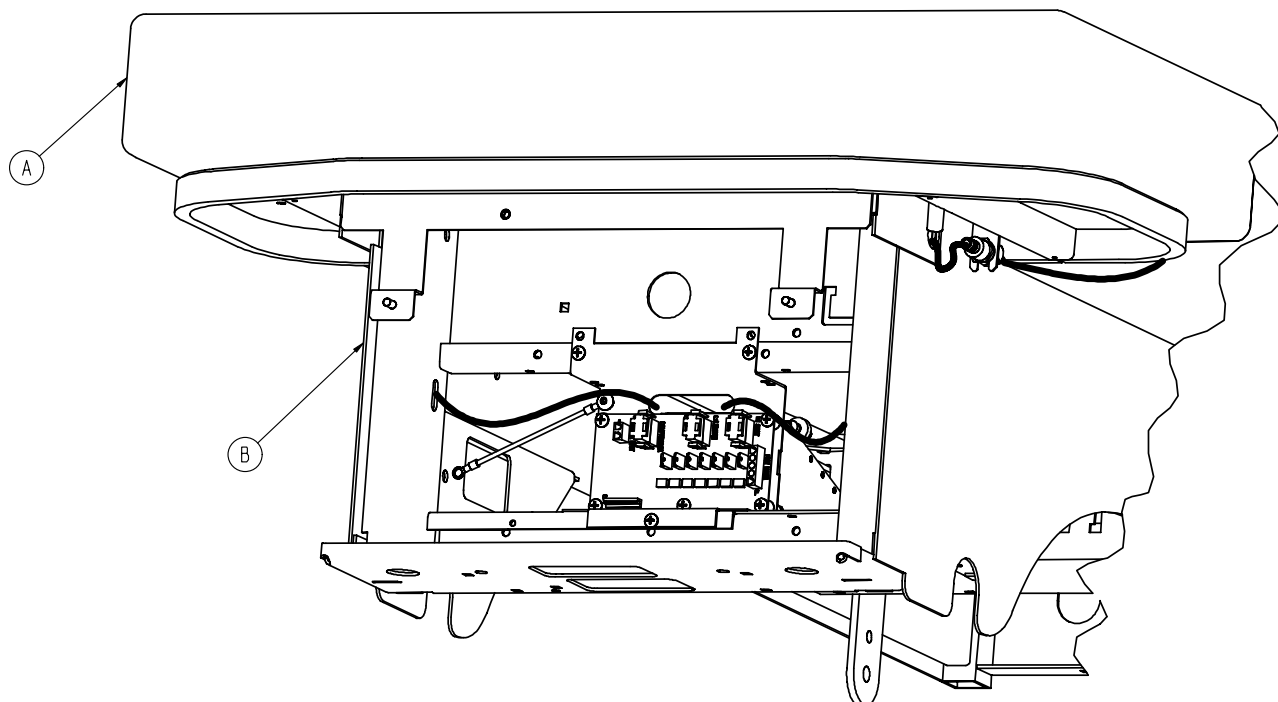

A	4701-045-000	1
	4701-045-010	1
B	4701-045-100	1

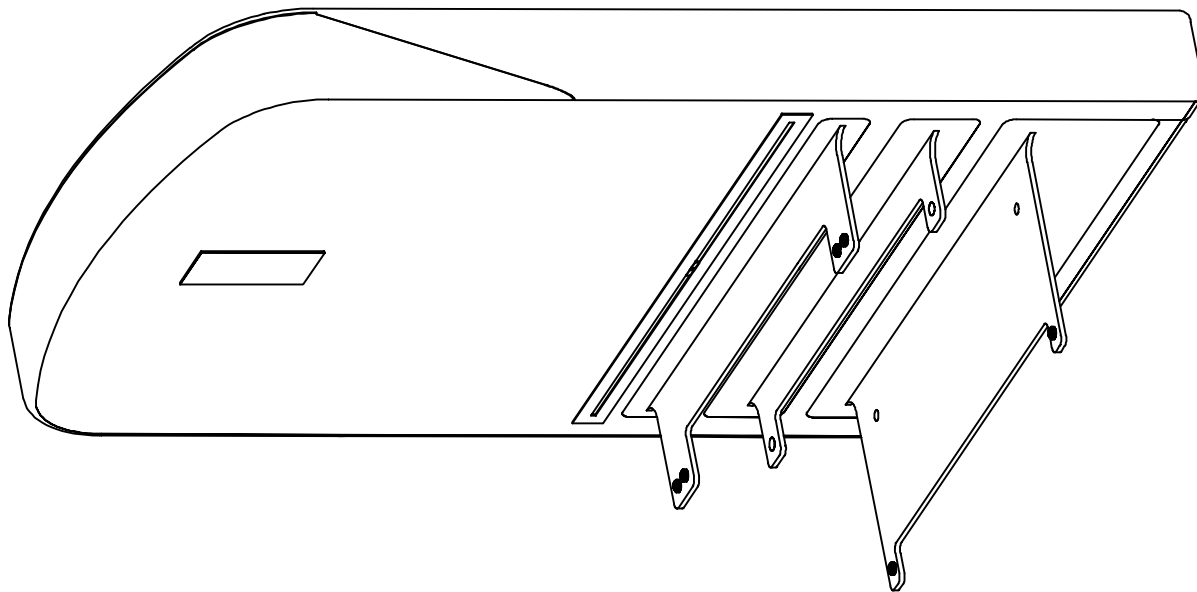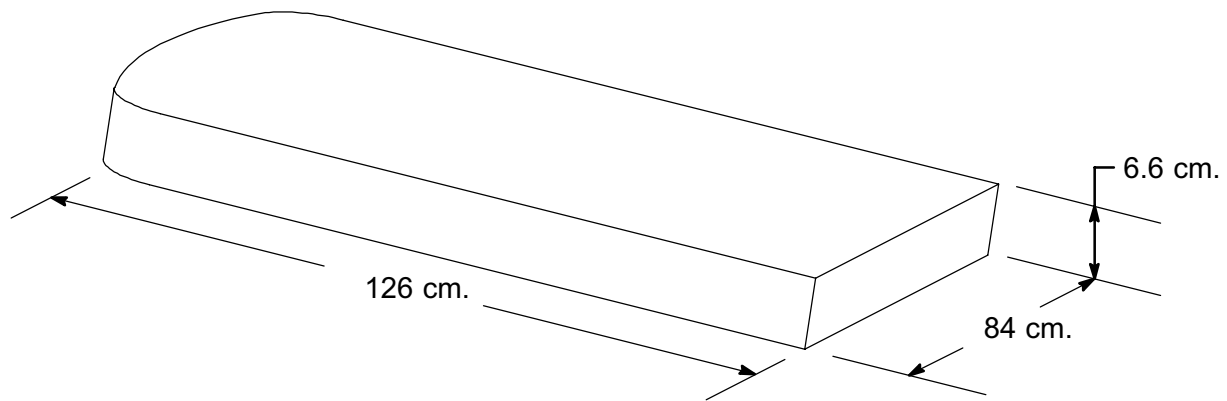


Rev C

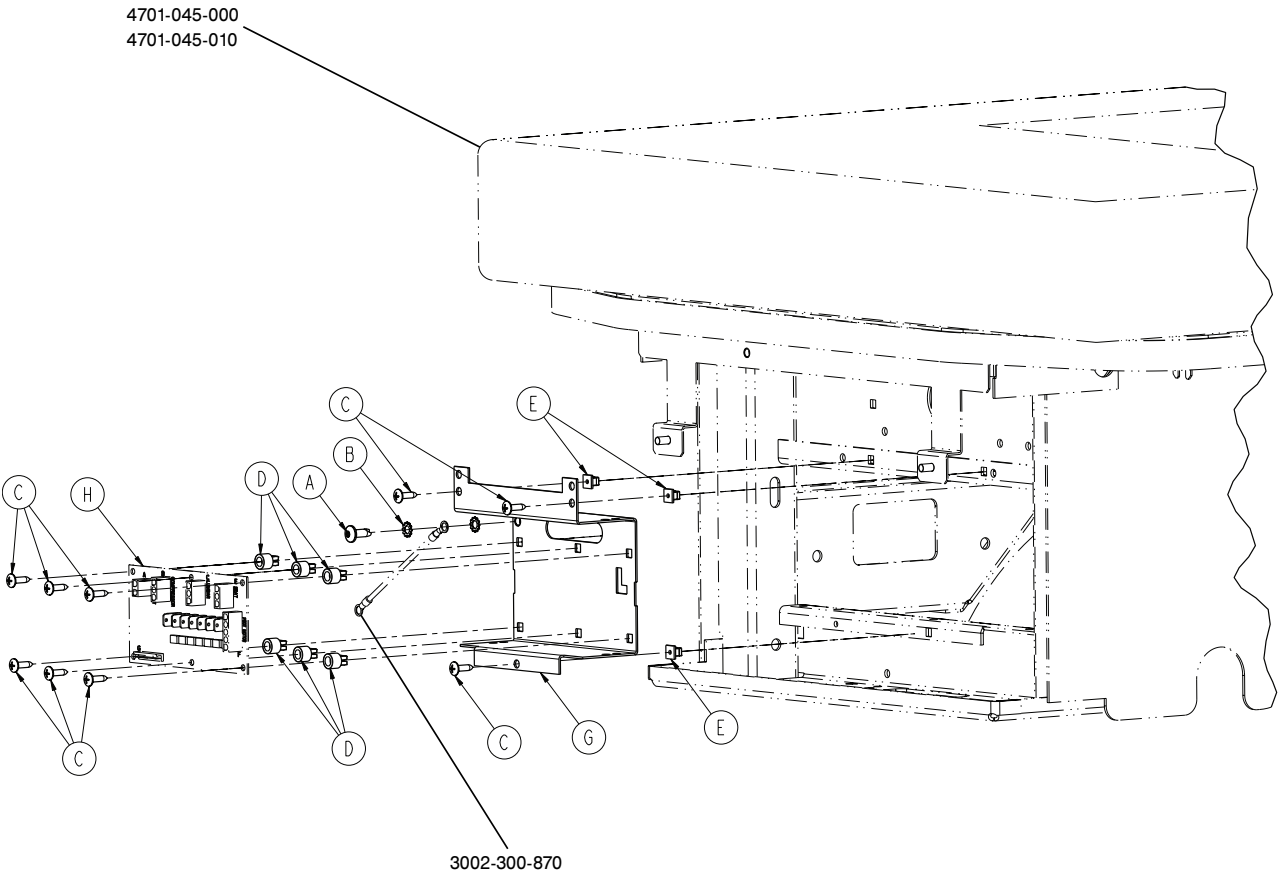


A	0007-065-000	1
B	0013-010-000	2
C	0023-080-000	9
D	0052-907-000	6
E	3000-300-002	3
G	4701-048-009	1
H	4701-080-050	1


Rev A

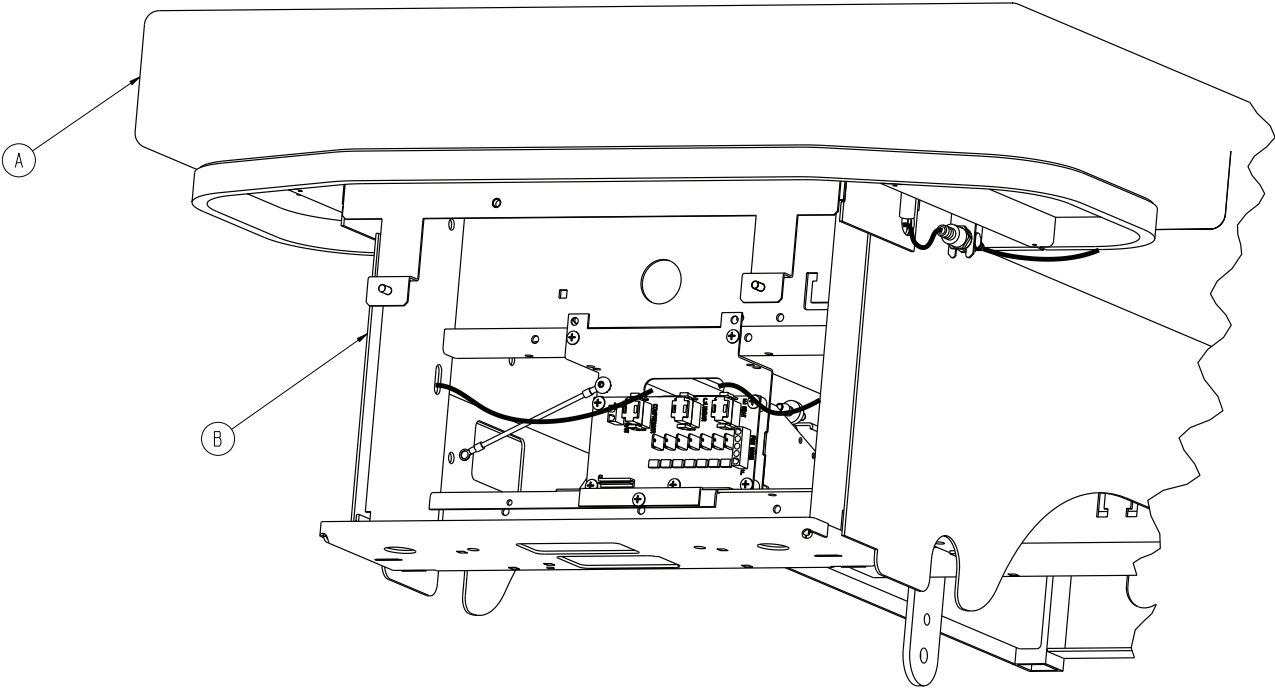

A	4701-048-000	1
	4701-048-020	1
B	4712-048-100	1

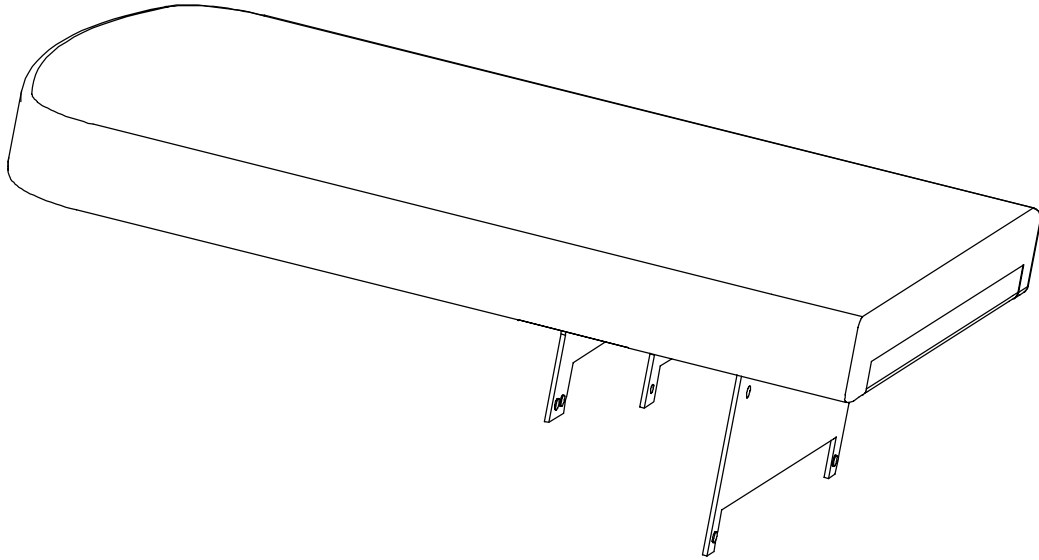



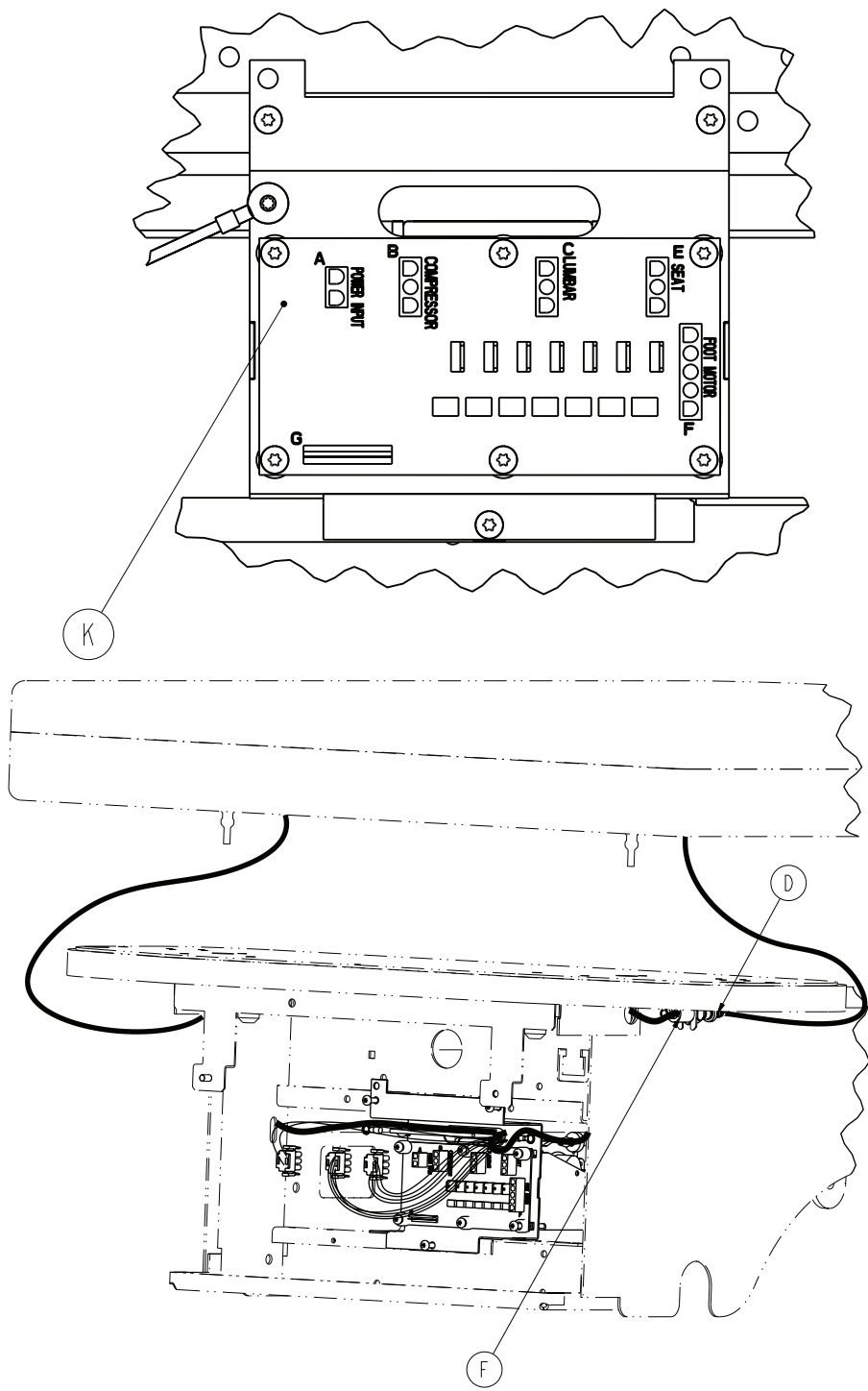



A	0007-065-000	1
B	0013-010-000	2
C	0023-080-000	9
D	0048-228-000	2
E	0052-907-000	6


4712-048-100

F	2500-900-012	2
G	3000-300-002	3
J	4712-048-030	1
K	4701-080-050	1
L	0059-086-000	1

Rev E

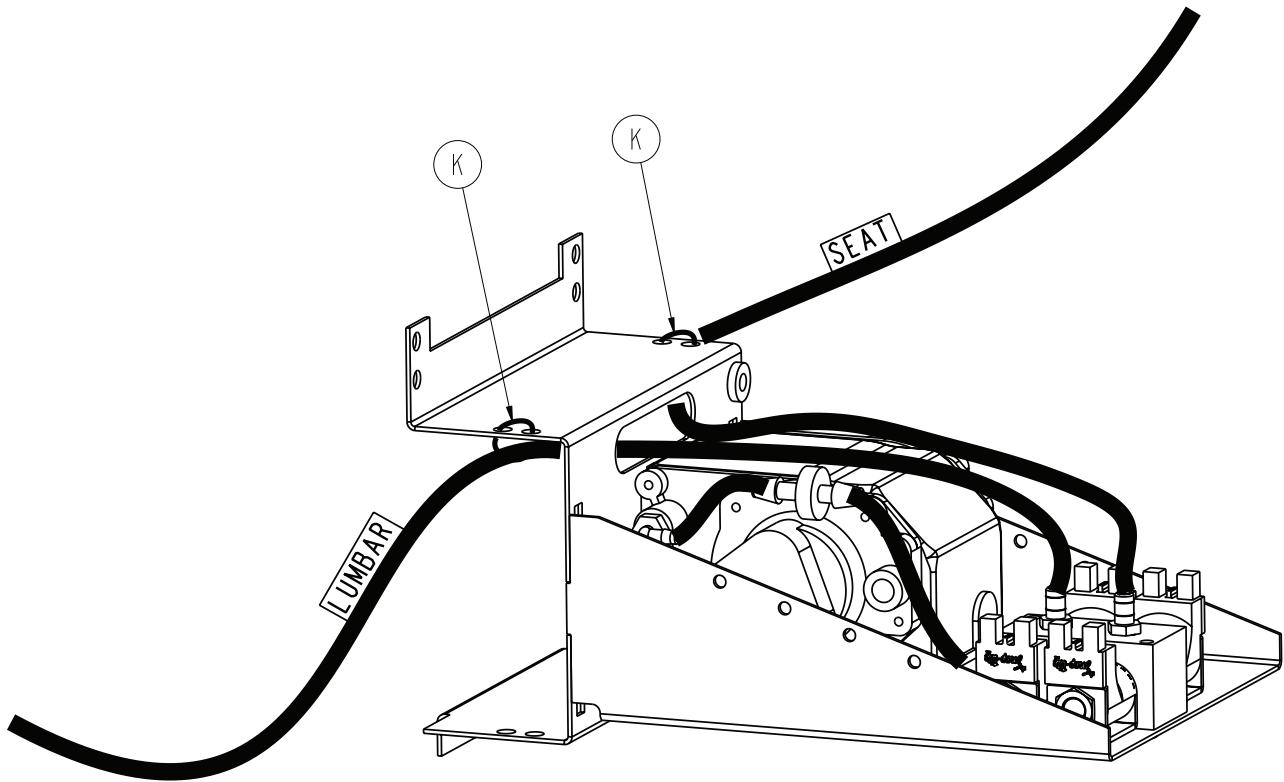


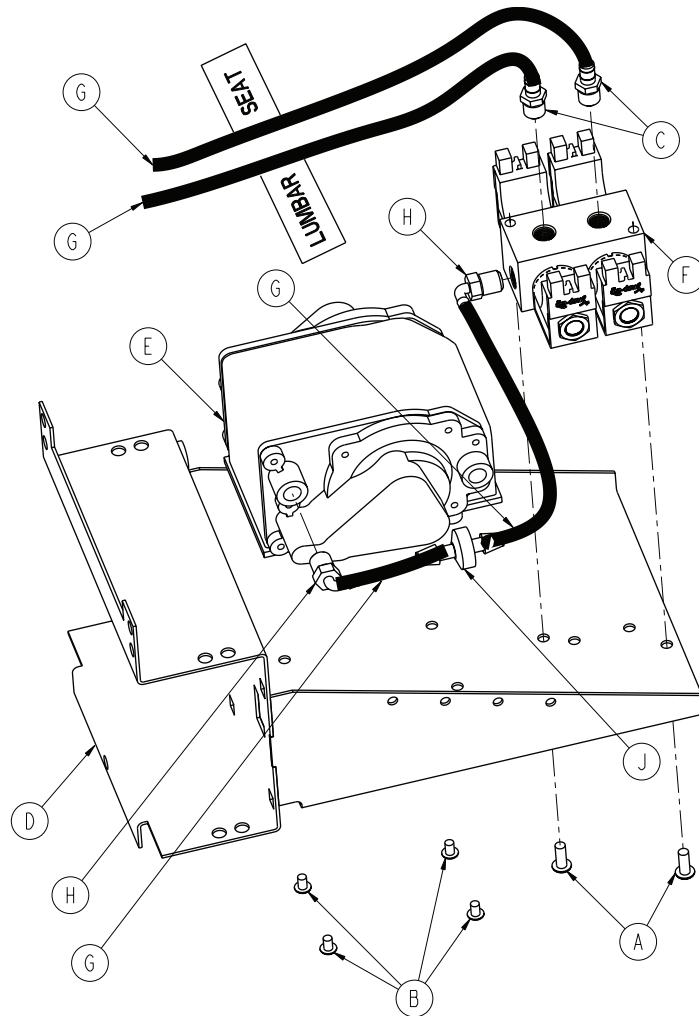


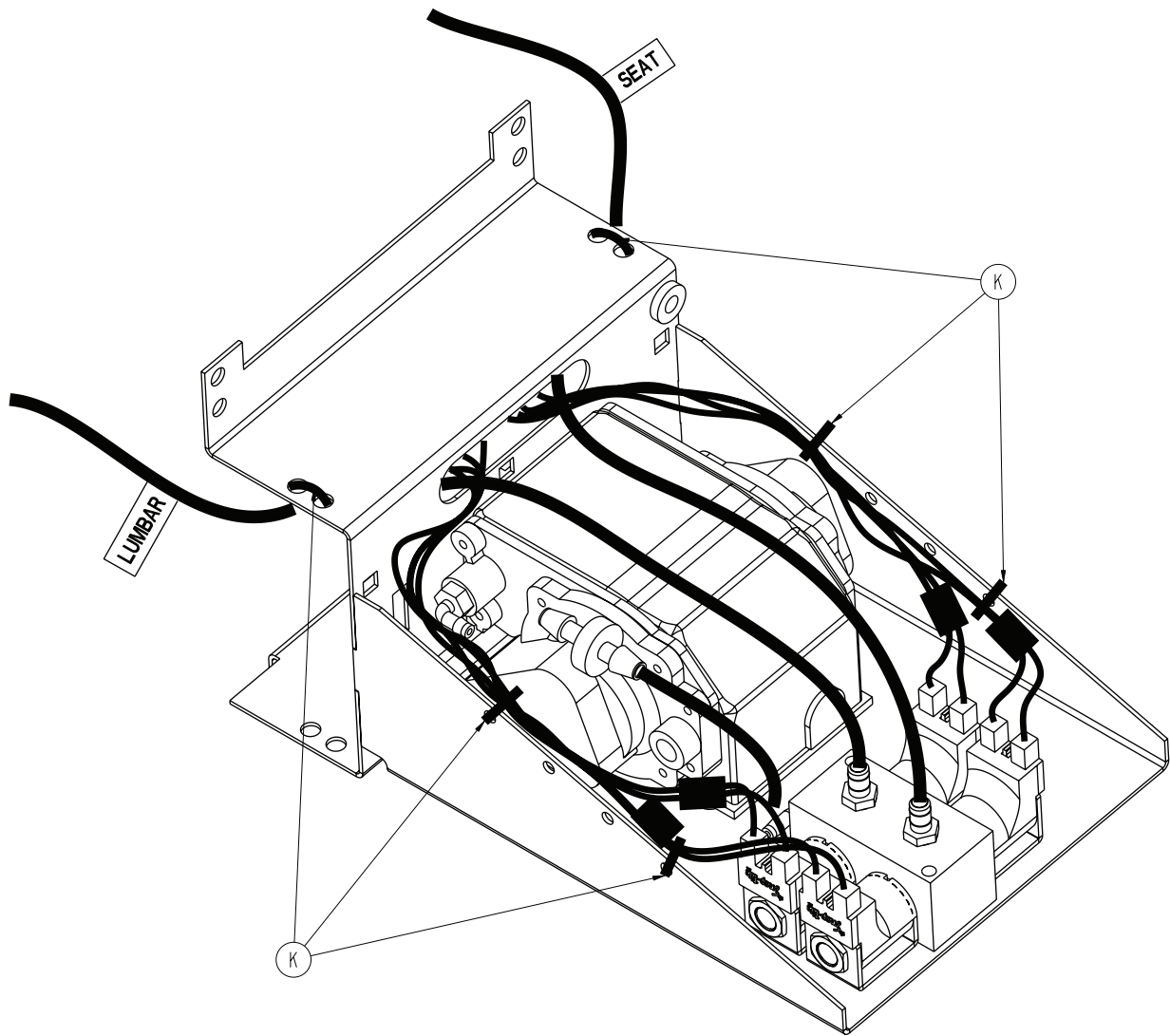

A	0004-304-000	2
B	0004-299-000	4
C	0048-159-000	2
D	4701-048-010	1
E	4712-048-015	1
F	4712-048-025	1
G	5002-030-240	4.58'
H	AP081227KM25N	2
J	AP19CV0025SN	1
K	0059-086-000	6

Rev B

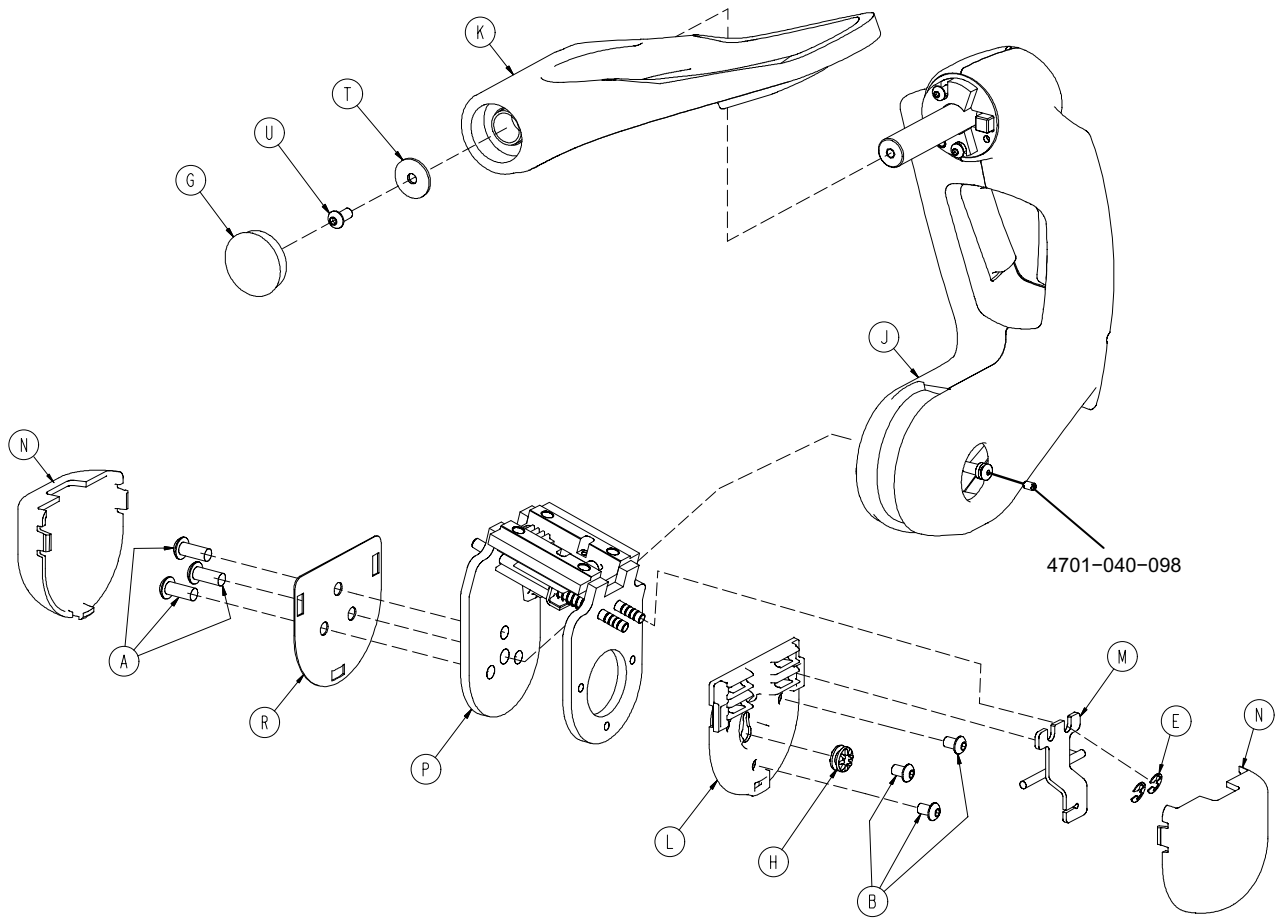

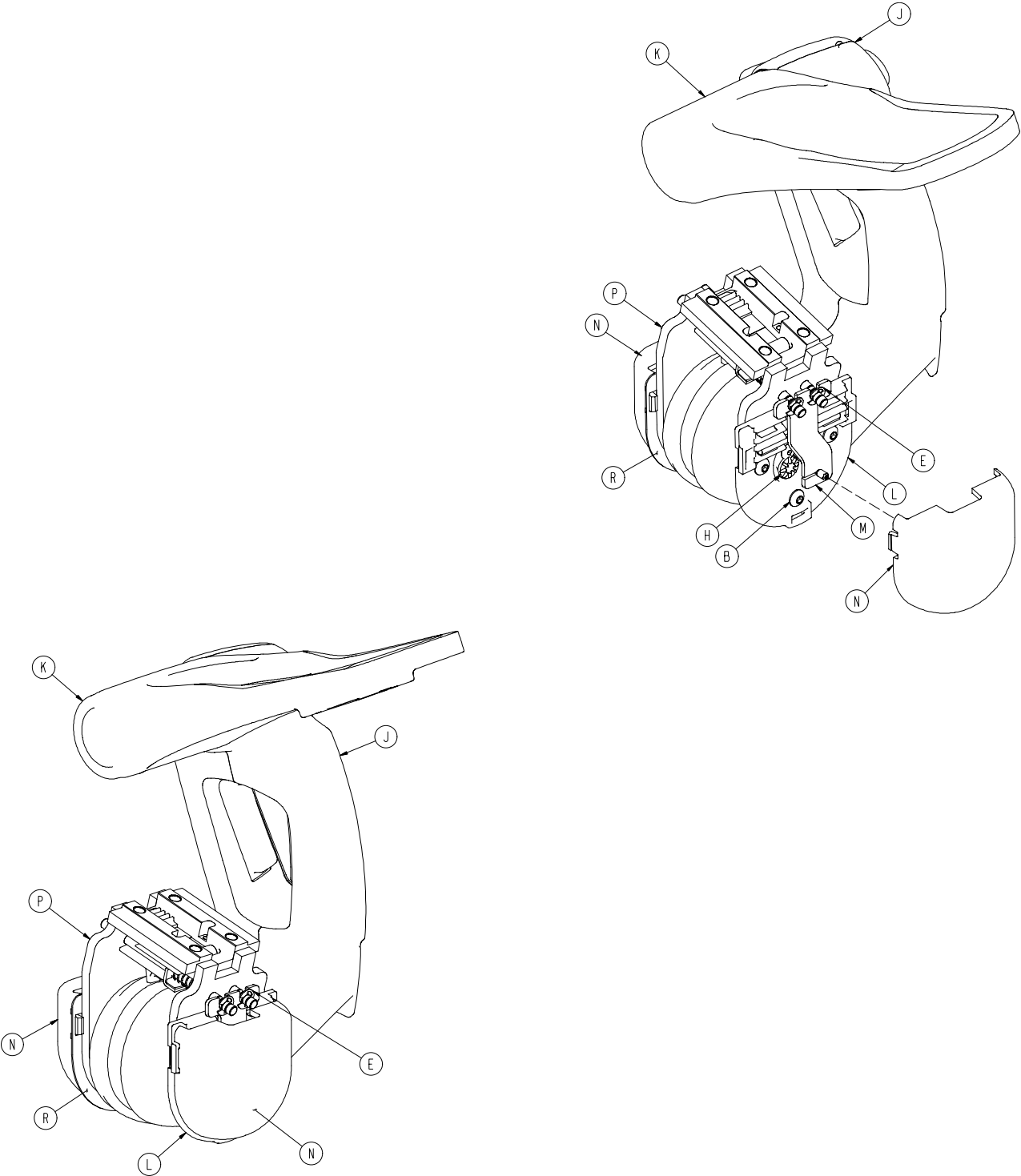



A	0004-582-000	3
B	0004-497-000	3
E	0028-114-000	2
G	0037-220-000	1

4701-040-070

H	3000-300-442	1
J	4701-040-050	1
K	4701-040-055	1
L	4701-040-078	1
M	4701-040-097	1
N	4701-040-099	2
P	4701-040-175	1
R	4701-040-158	1
T	0011-053-000	1
U	0004-325-000	1

Rev B

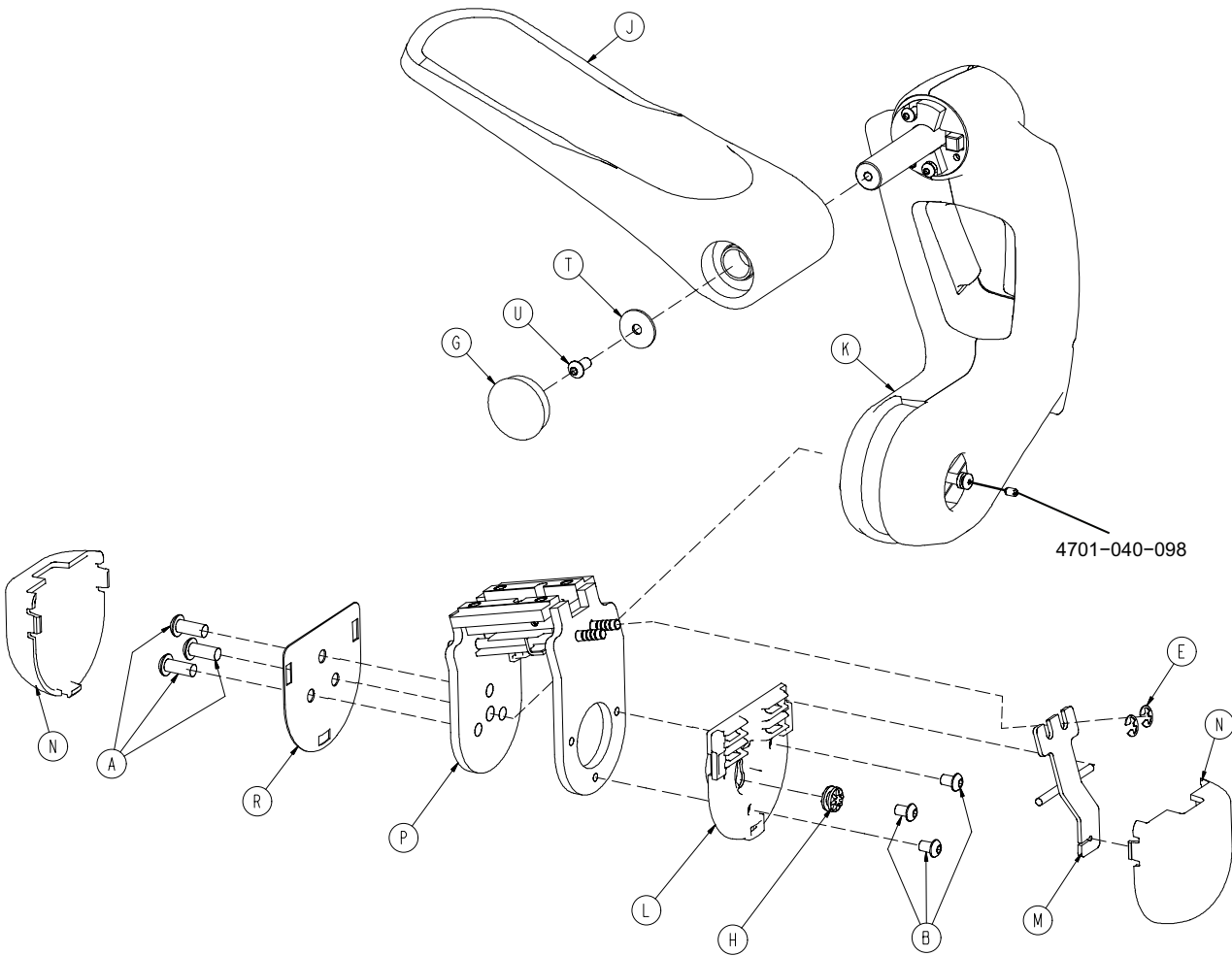


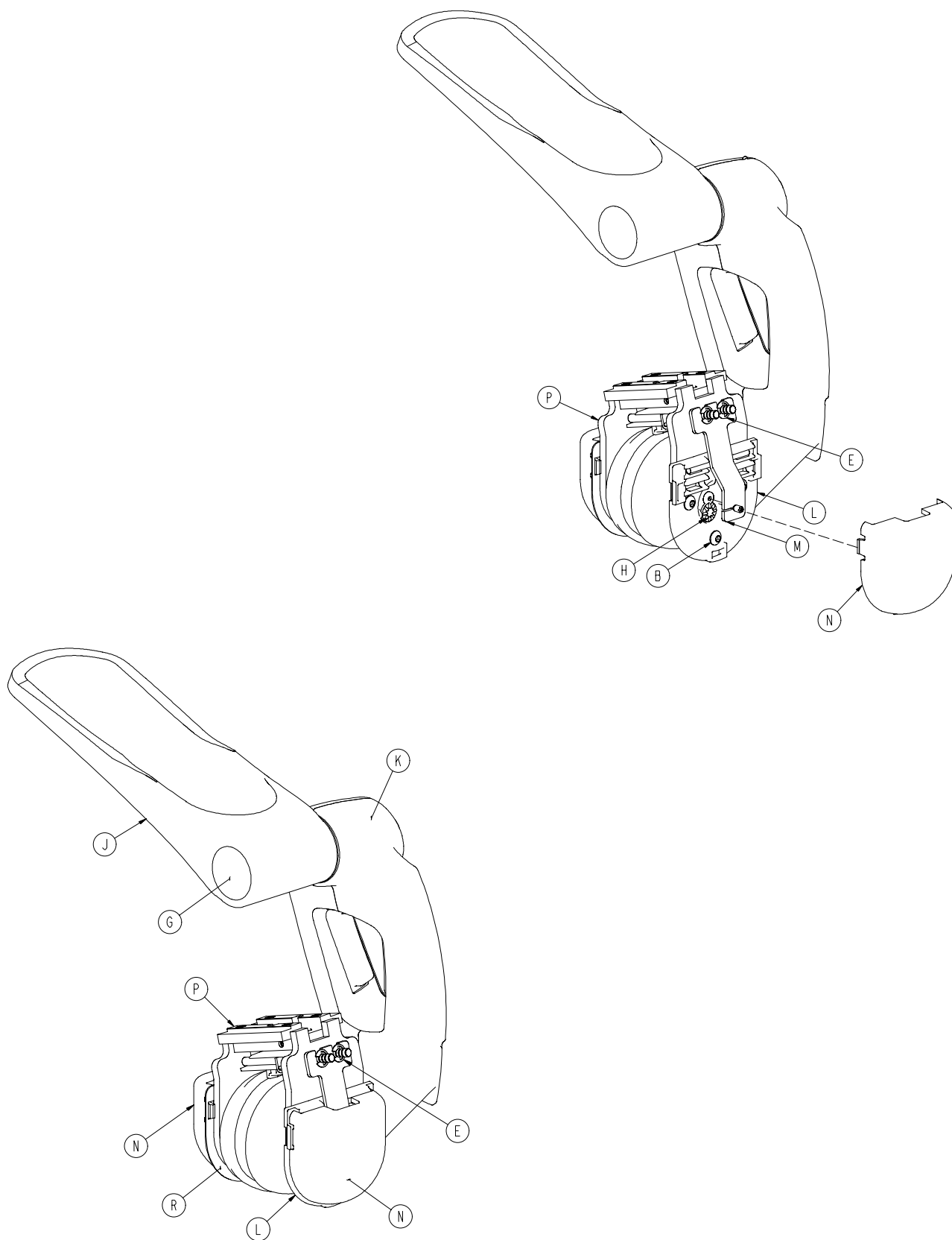

4701-040-060


A	0004-582-000	3
B	0004-497-000	3
E	0028-114-000	2
G	0037-220-000	1
H	3000-300-442	1
J	4701-040-045	1
K	4701-040-050	1
L	4701-040-078	1
M	4701-040-096	1
N	4701-040-099	2
P	4701-040-180	1
R	4701-040-158	1
T	0011-053-000	1
U	0004-325-000	1

Rev B/Rev B

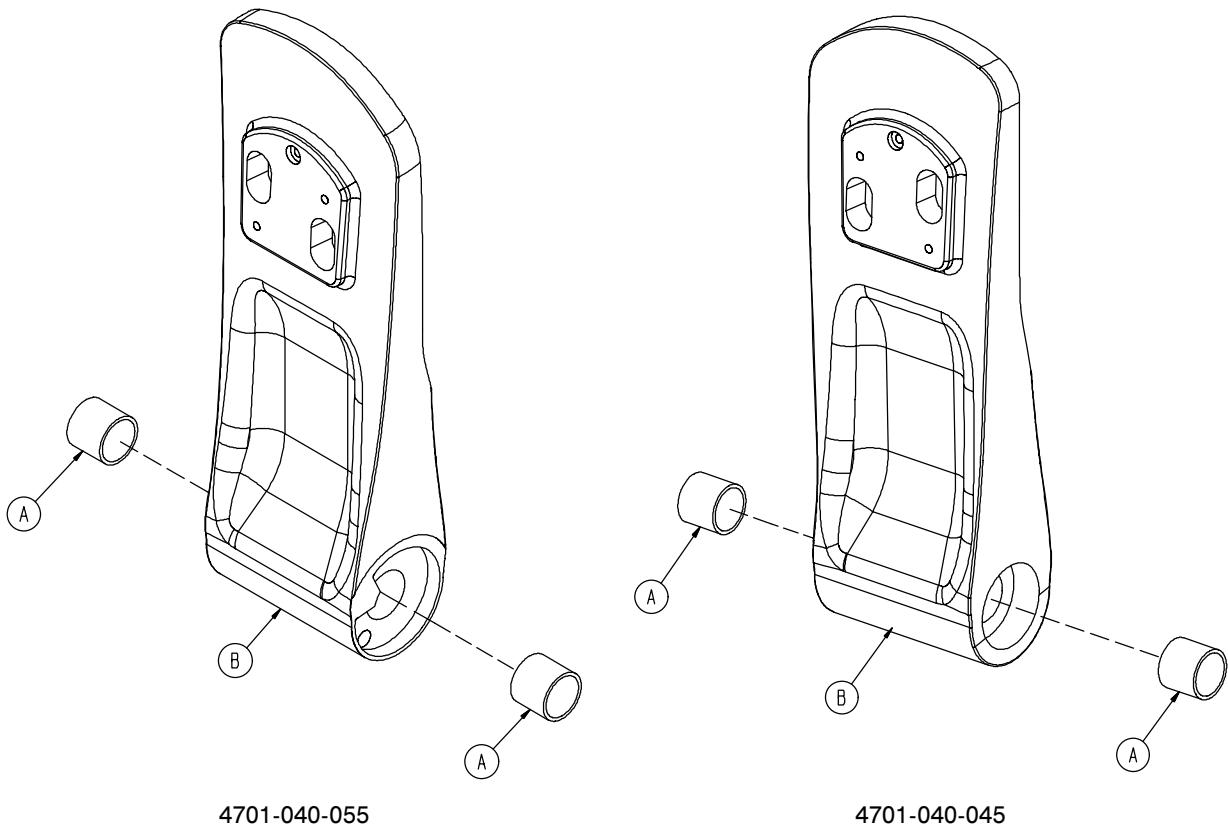


A	0081-289-000	2
B	4701-040-072	1
	4701-040-073	1

Rev B

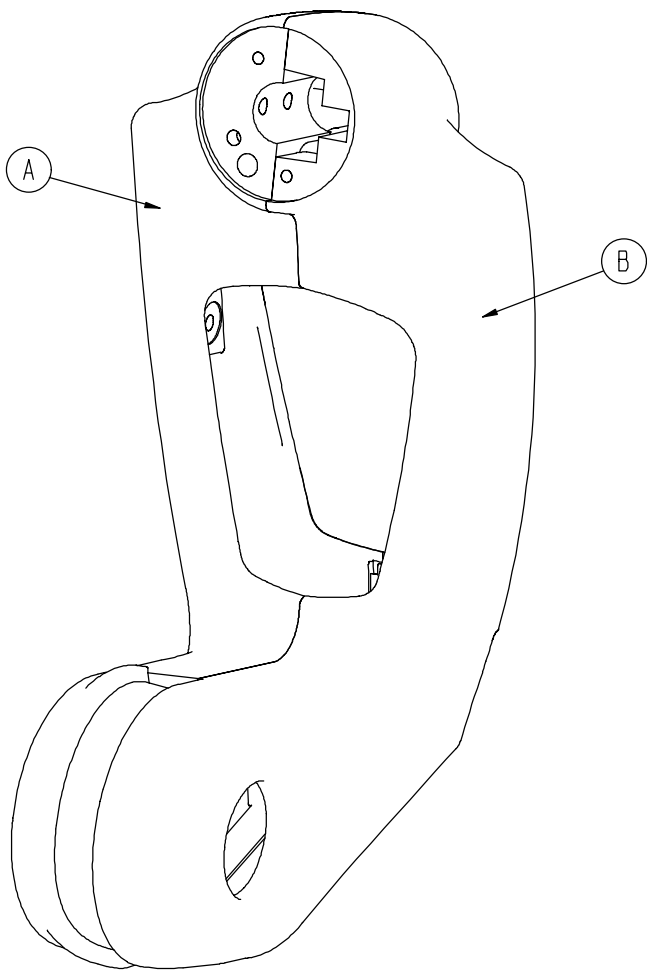


A	4701-040-043	1
B	4701-040-044	1

Rev C

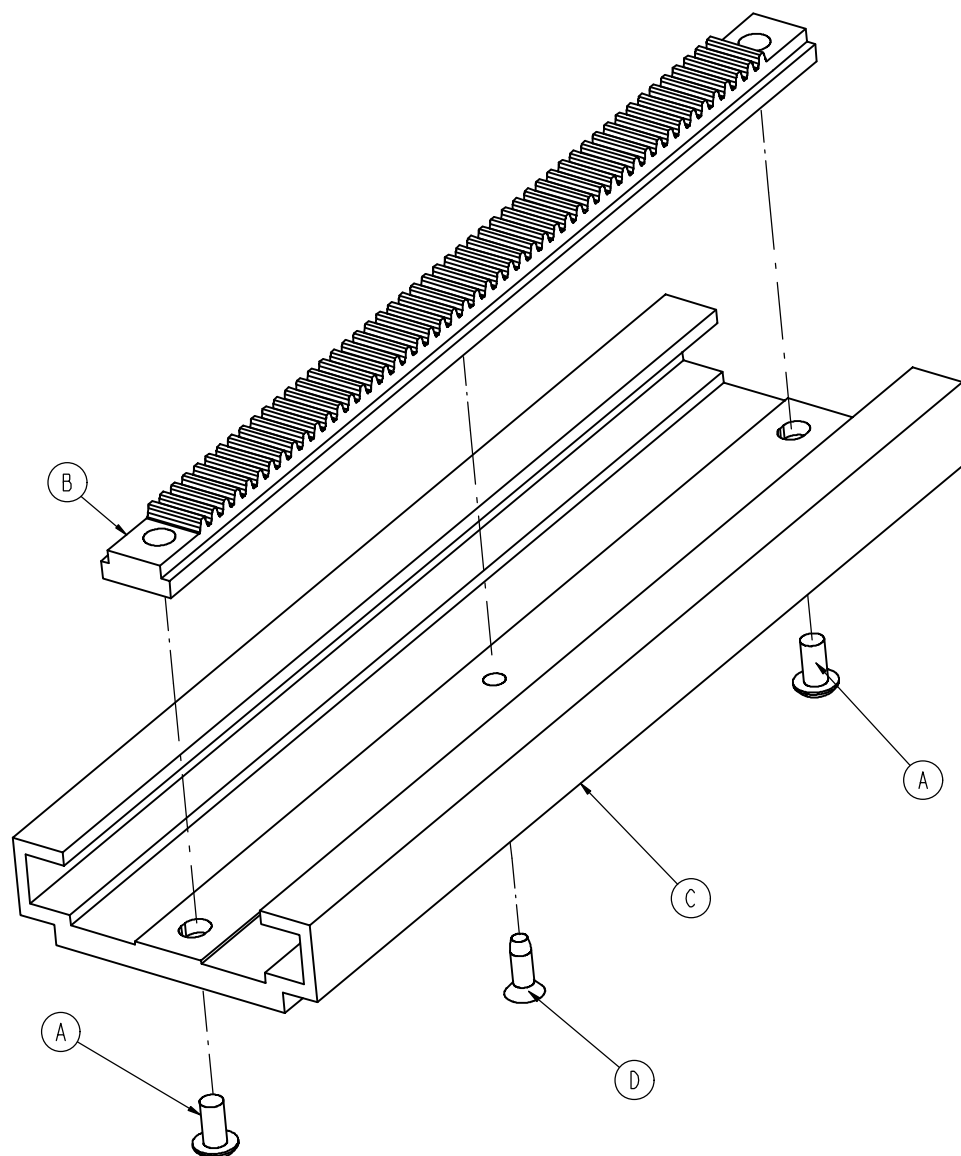


A	0004-375-000	2
B	4701-040-079	1
C	4701-040-034	1
D	0023-280-000	1

Rev B

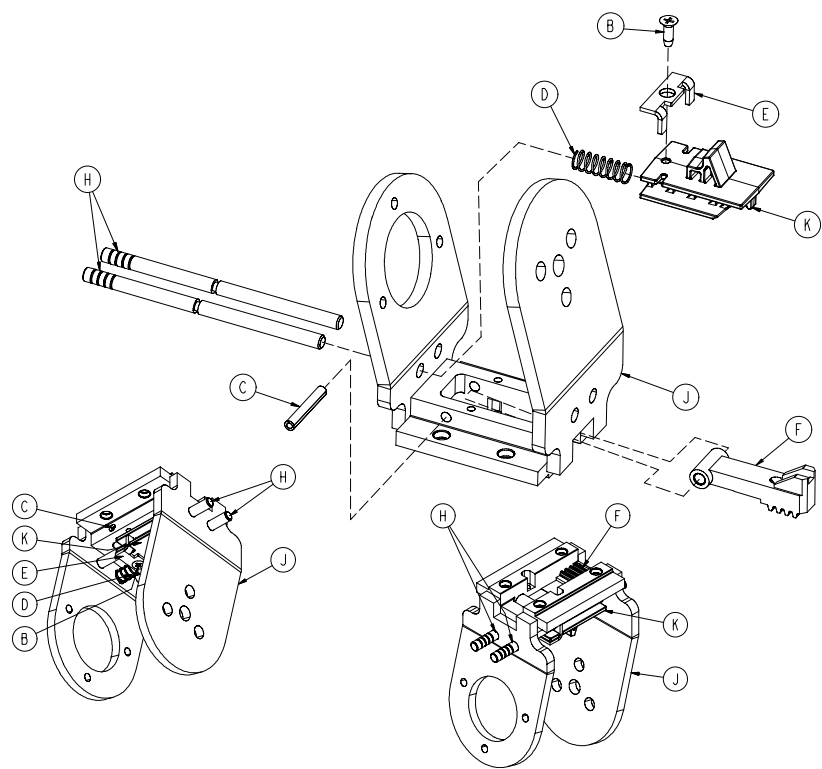


B	0023-280-000	1
C	0026-012-000	1
D	0038-558-000	1
E	4701-040-071	1
F	4701-040-077	1
H	4701-040-089	2
J	4701-040-090	1
K	4701-040-093	1

Rev B

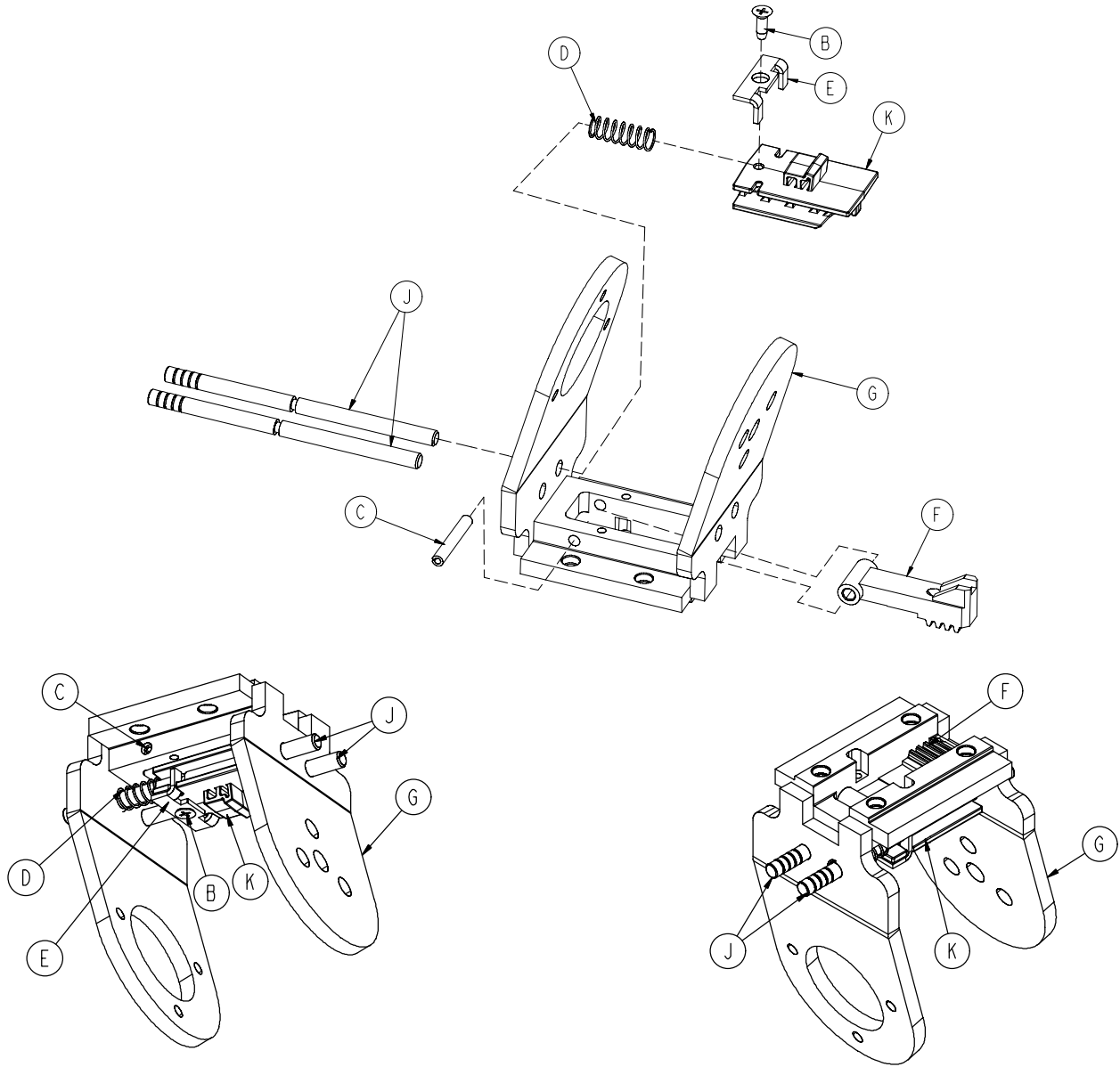

B	0023-280-000	1
C	0026-012-000	1
D	0038-558-000	1
E	4701-040-071	1
F	4701-040-077	1
G	4701-040-080	1
J	4701-040-089	2
K	4701-040-094	1

Rev A

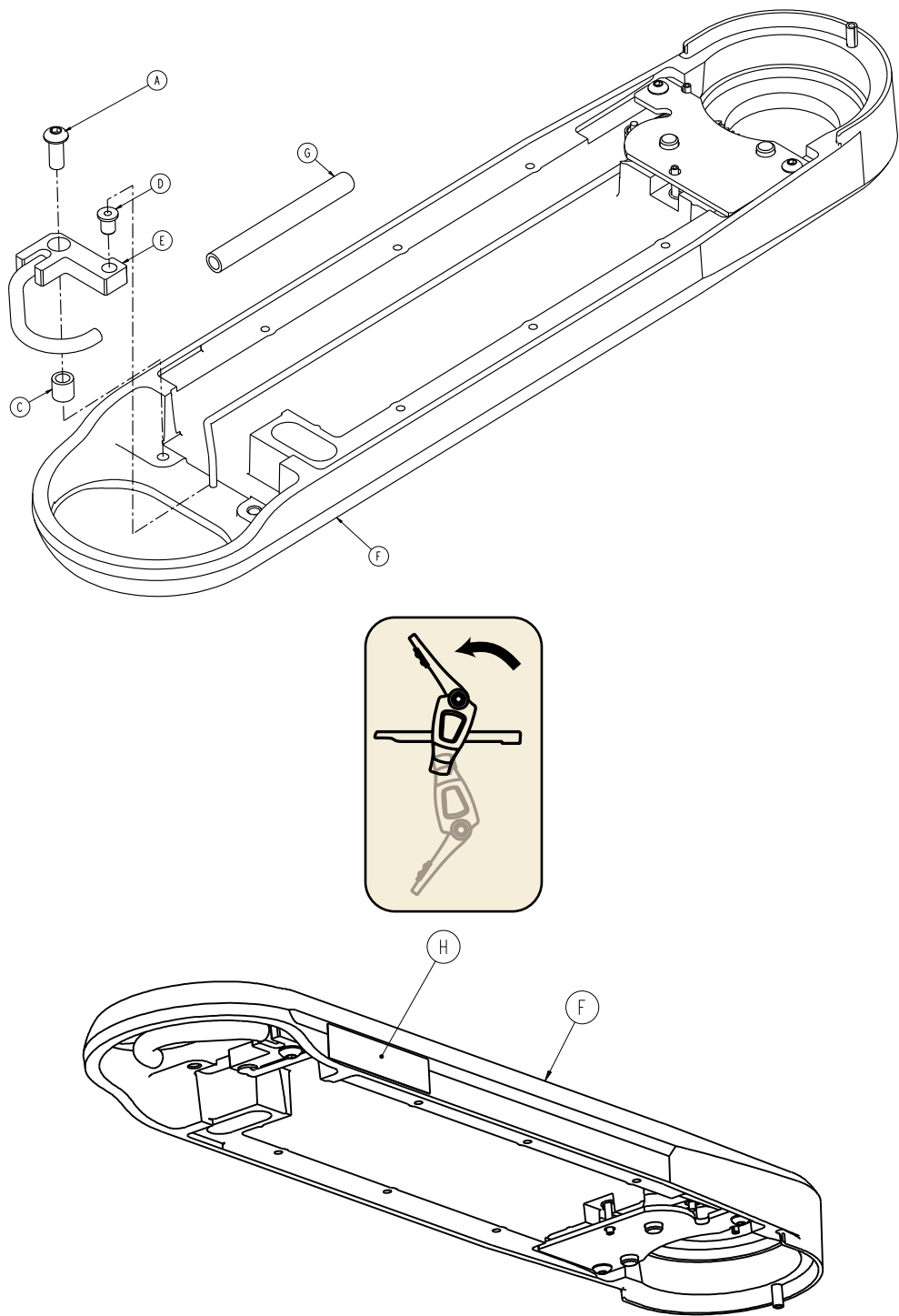


A	0004-509-000	1
C	0081-022-000	1
D	0081-436-000	1

4701-140-165


E	4701-040-145	1
F	4701-040-160	1
G	4701-040-153	1
H	4701-109-941	1


Rev A

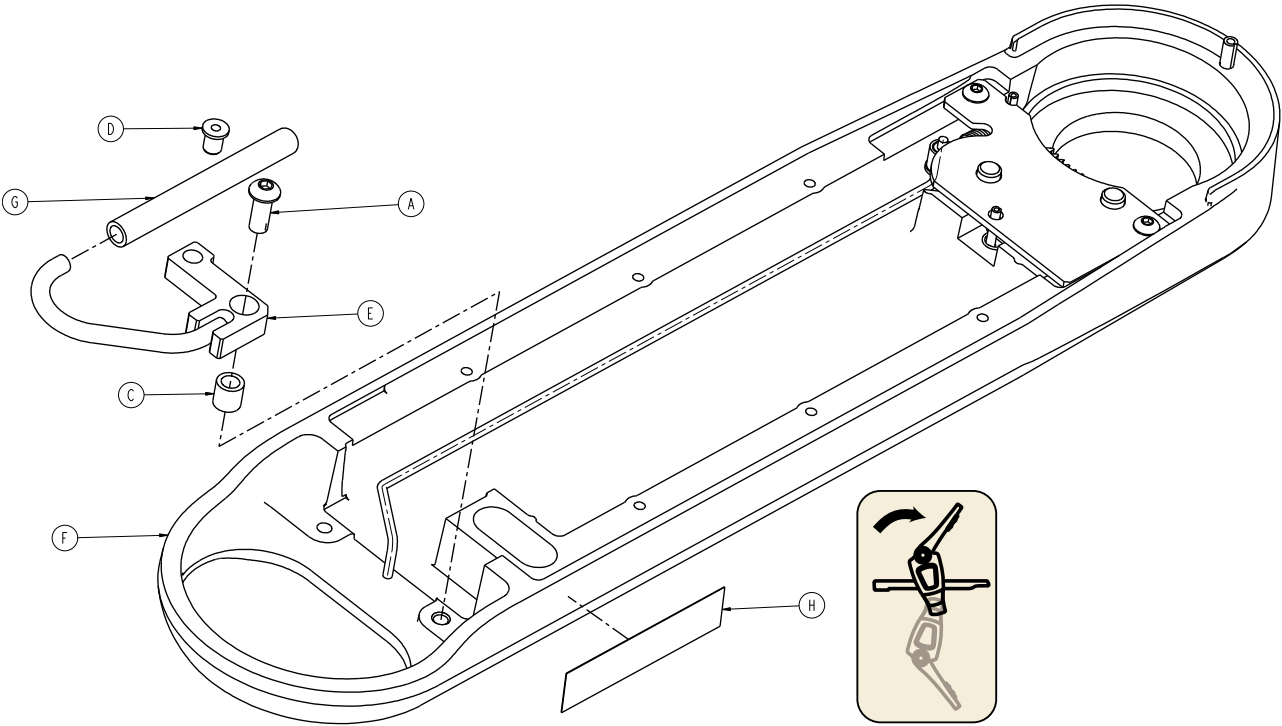

A	0004-509-000	1
C	0081-022-000	1
D	0081-436-000	1
E	4701-040-145	1
F	4701-040-160	1
G	4701-040-153	1
H	4701-109-942	1



Rev D

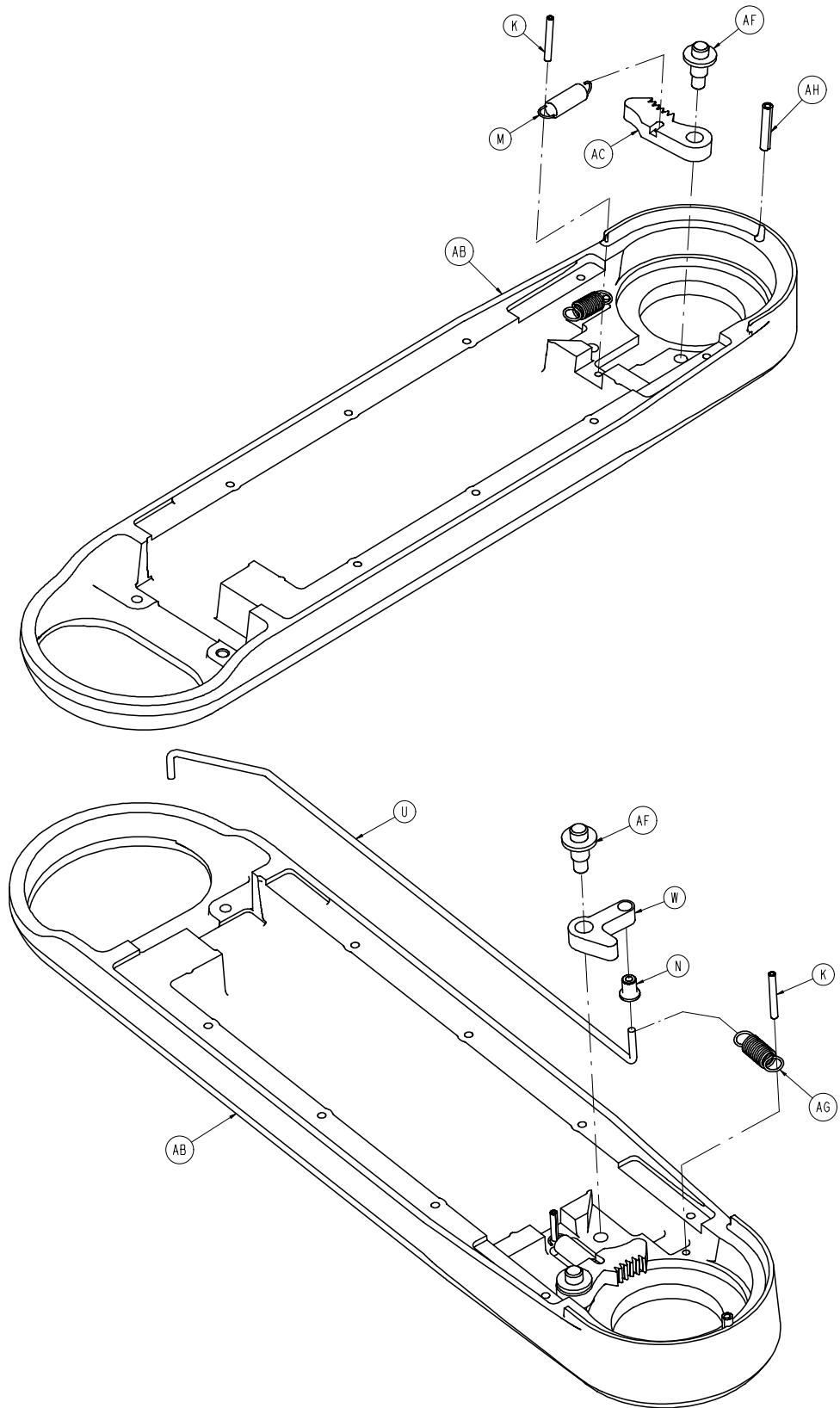

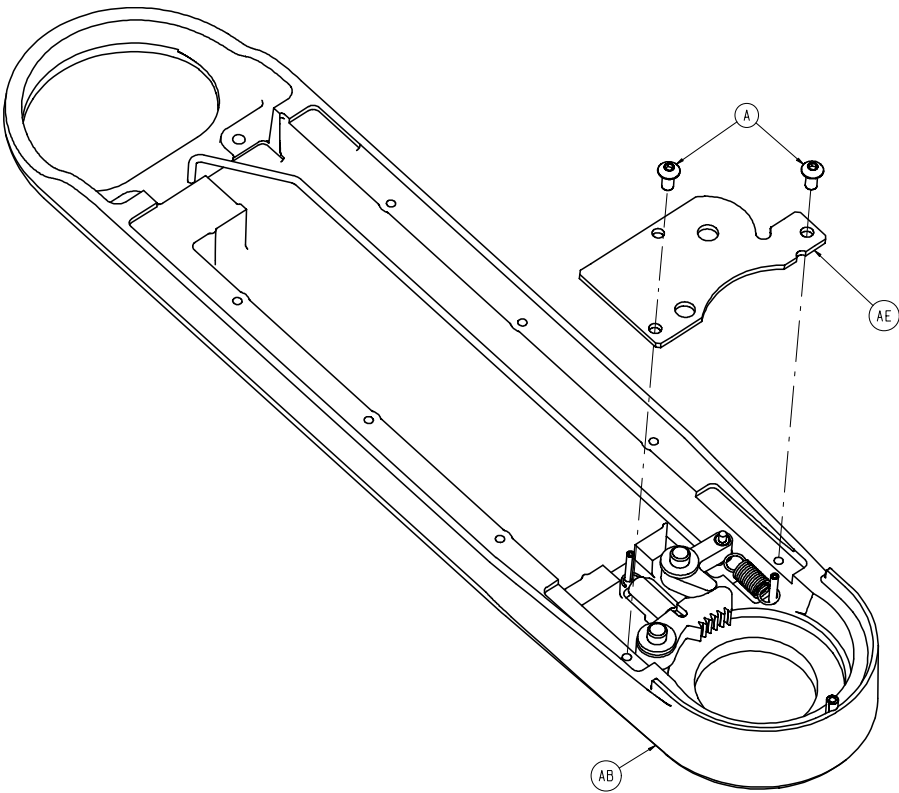


A	0004-375-000	2
K	0026-007-000	2
M	0038-458-000	2
N	0081-436-000	1
U	4701-040-036	1
W	4701-040-039	1
AB	4701-040-056	1
AC	4701-040-114	1
AE	4701-040-134	1
AF	4701-040-149	2
AG	0038-562-000	1
AH	0026-313-000	1

Rev B

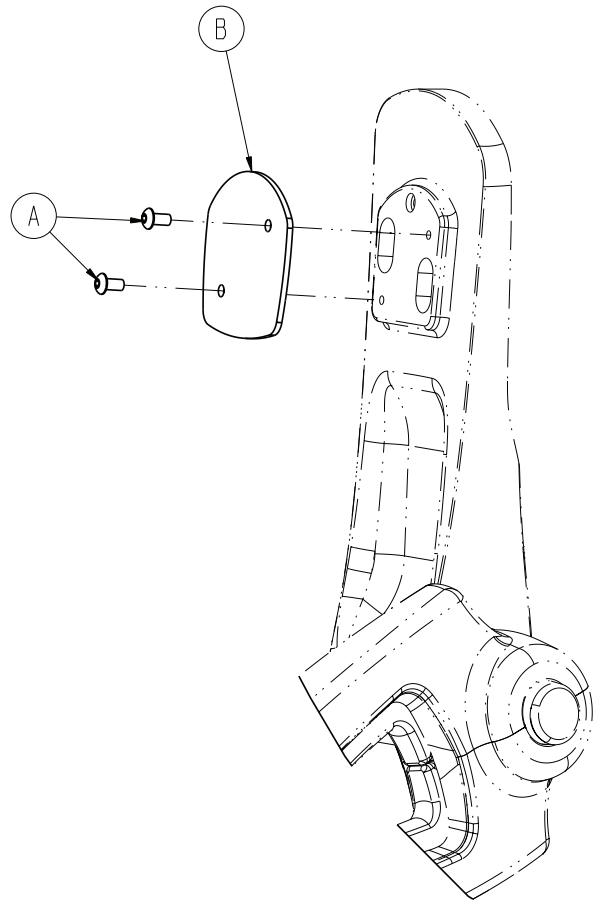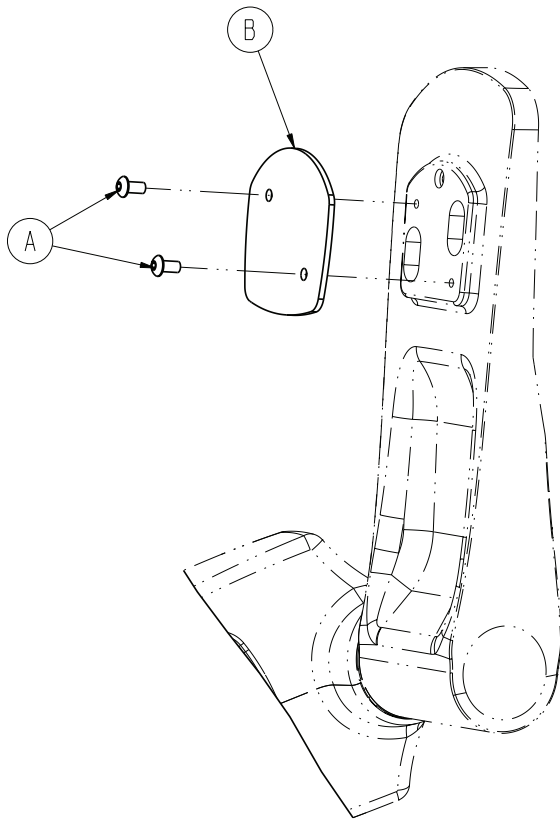



A	0004-375-000	4
B	4701-040-128	2

Rev A

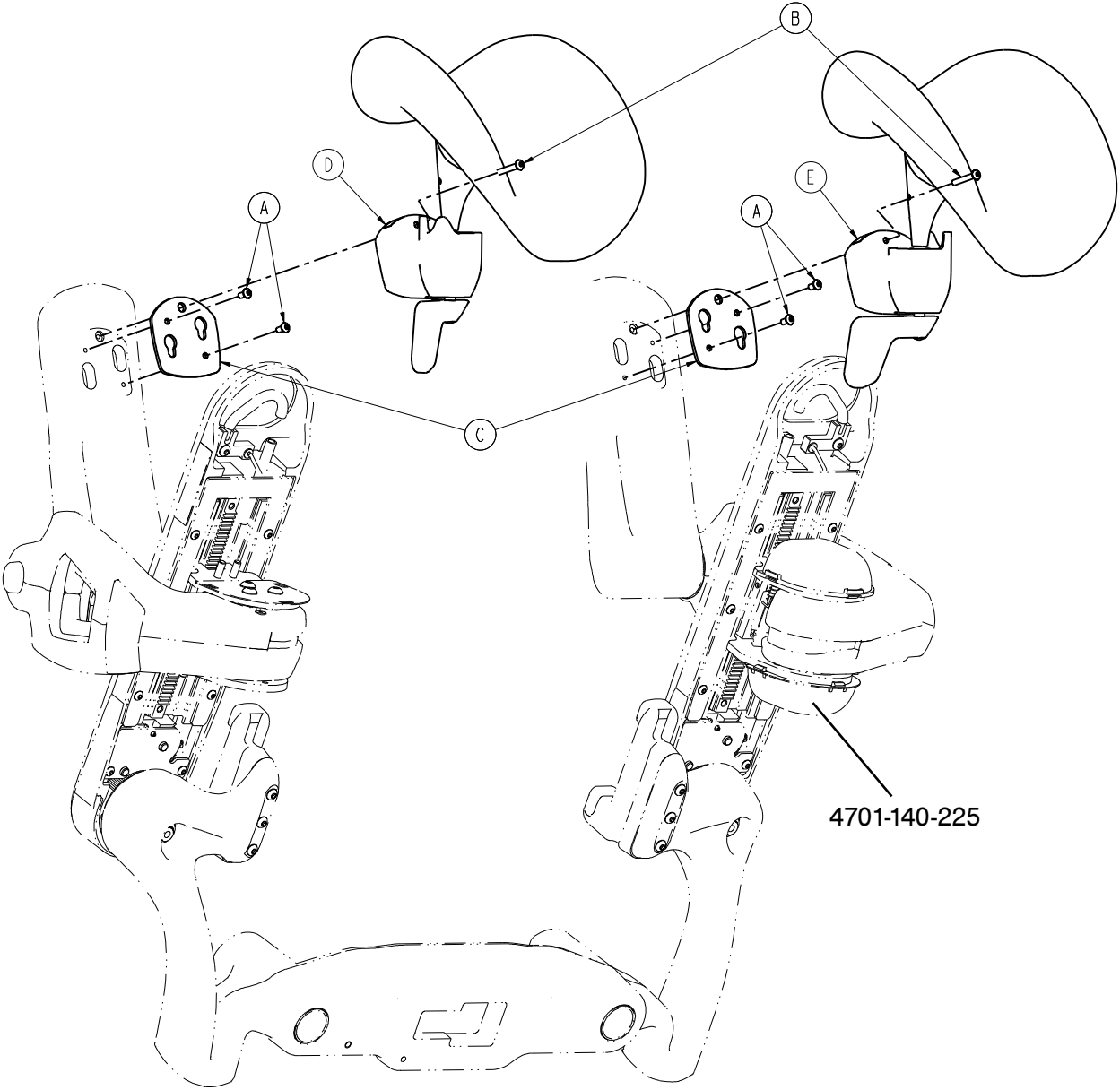

A	0004-375-000	4
B	0004-601-000	2
C	4701-040-119	2
D	4701-940-270	1
E	4701-940-275	1


Rev A

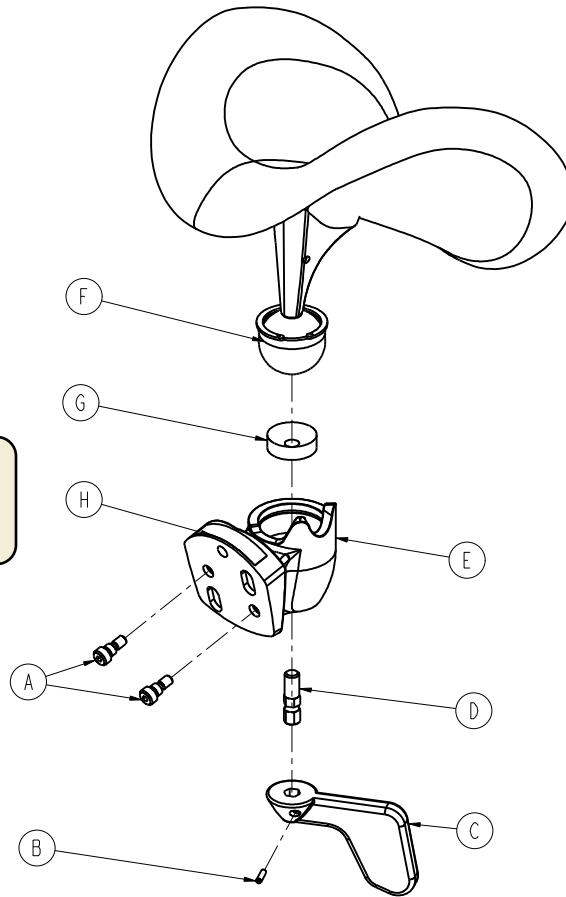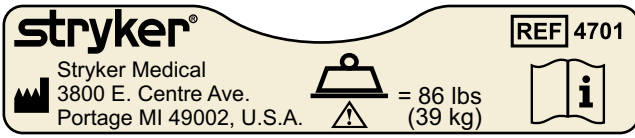


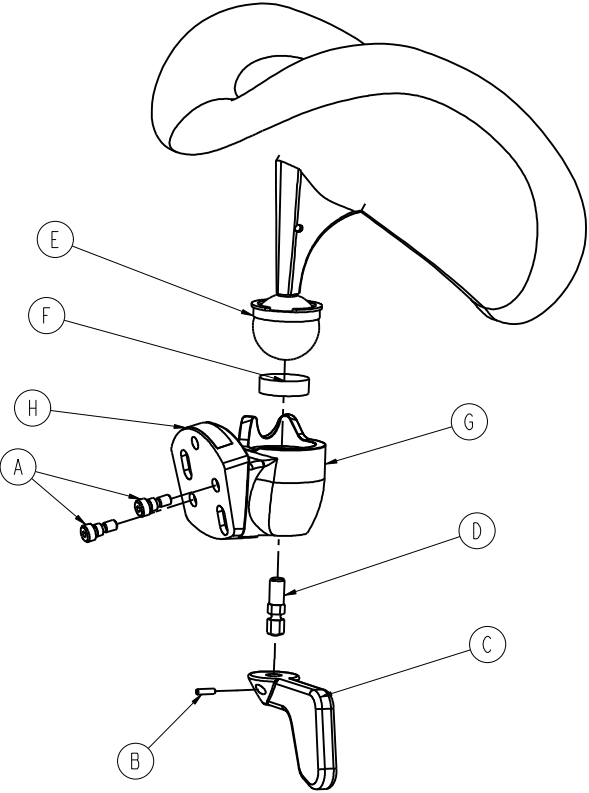
A	0008-062-000	2
B	0021-181-000	1
C	4701-840-170	1
D	4701-840-171	1
E	4701-840-211	1
F	4701-840-265	1
G	4701-840-271	1
H	4701-109-982	1

Rev A

stryker[®]
Stryker Medical
3800 E. Centre Ave.
Portage MI 49002, U.S.A.



= 86 lbs
(39 kg)

REF 4701




A	0008-062-000	2
B	0021-181-000	1
C	4701-840-170	1
D	4701-840-171	1
E	4701-840-265	1
F	4701-840-271	1
G	4701-840-311	1
H	4701-109-982	1


Rev A

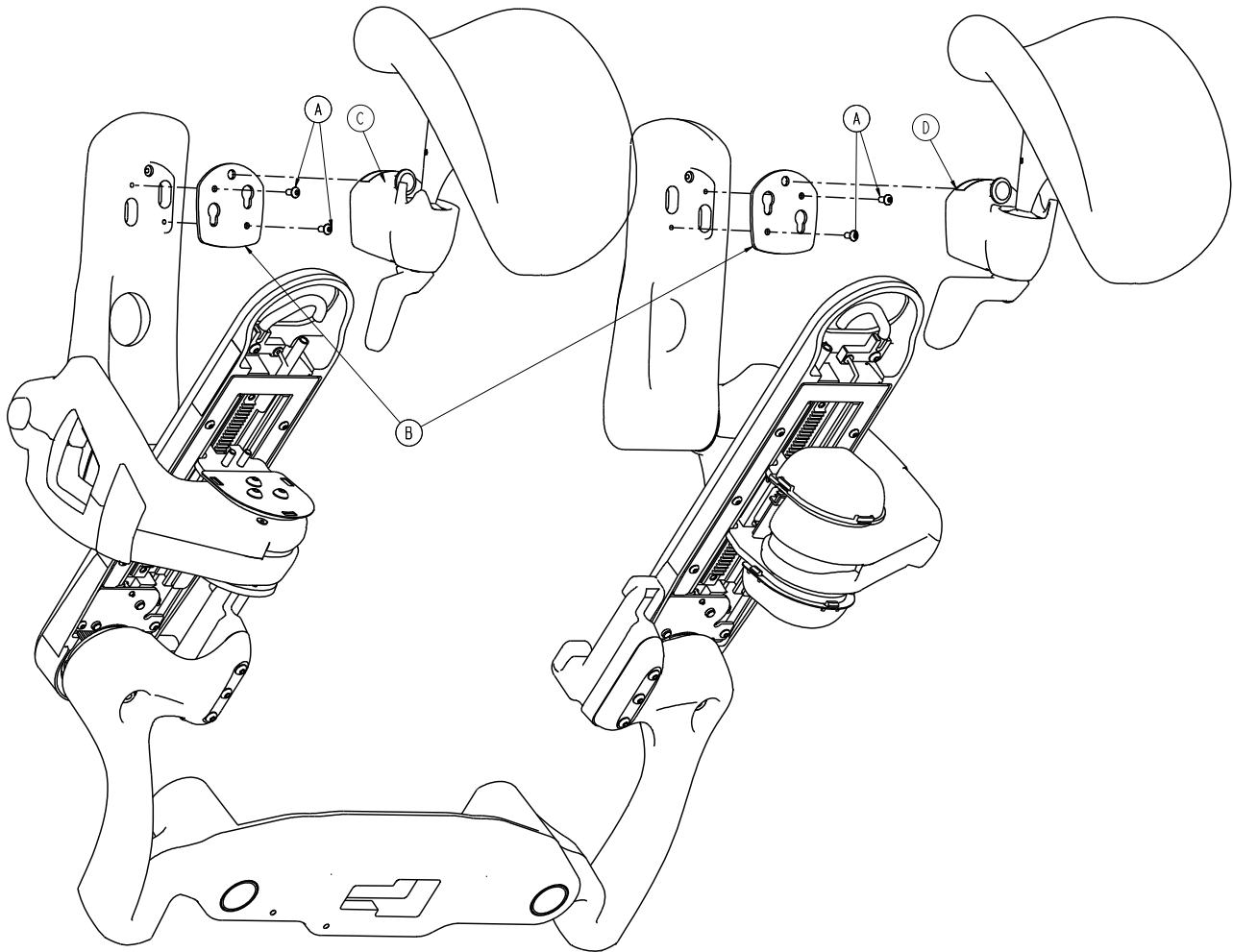


A	0004-375-000	4
B	4701-040-119	2
C	4701-940-280	1
D	4701-940-285	1

4701-940-280/4701-940-285

Rev A/Rev A

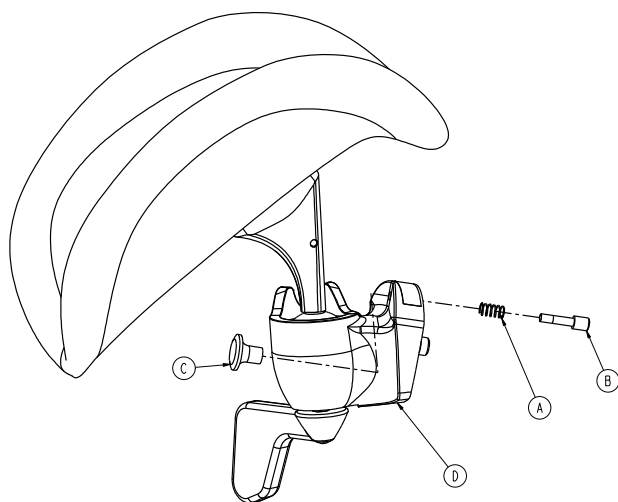


4701-940-280

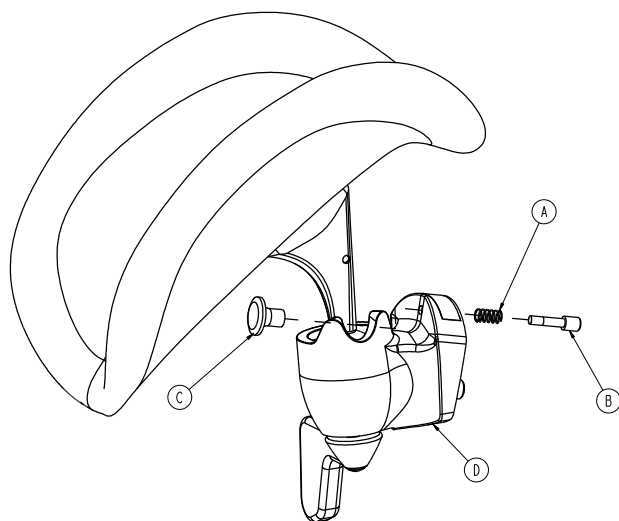

4701-940-285



A	0038-469-000	1
B	4701-040-126	1
C	4701-040-127	1
D	4701-940-270	1
	4701-940-275	1

Rev A

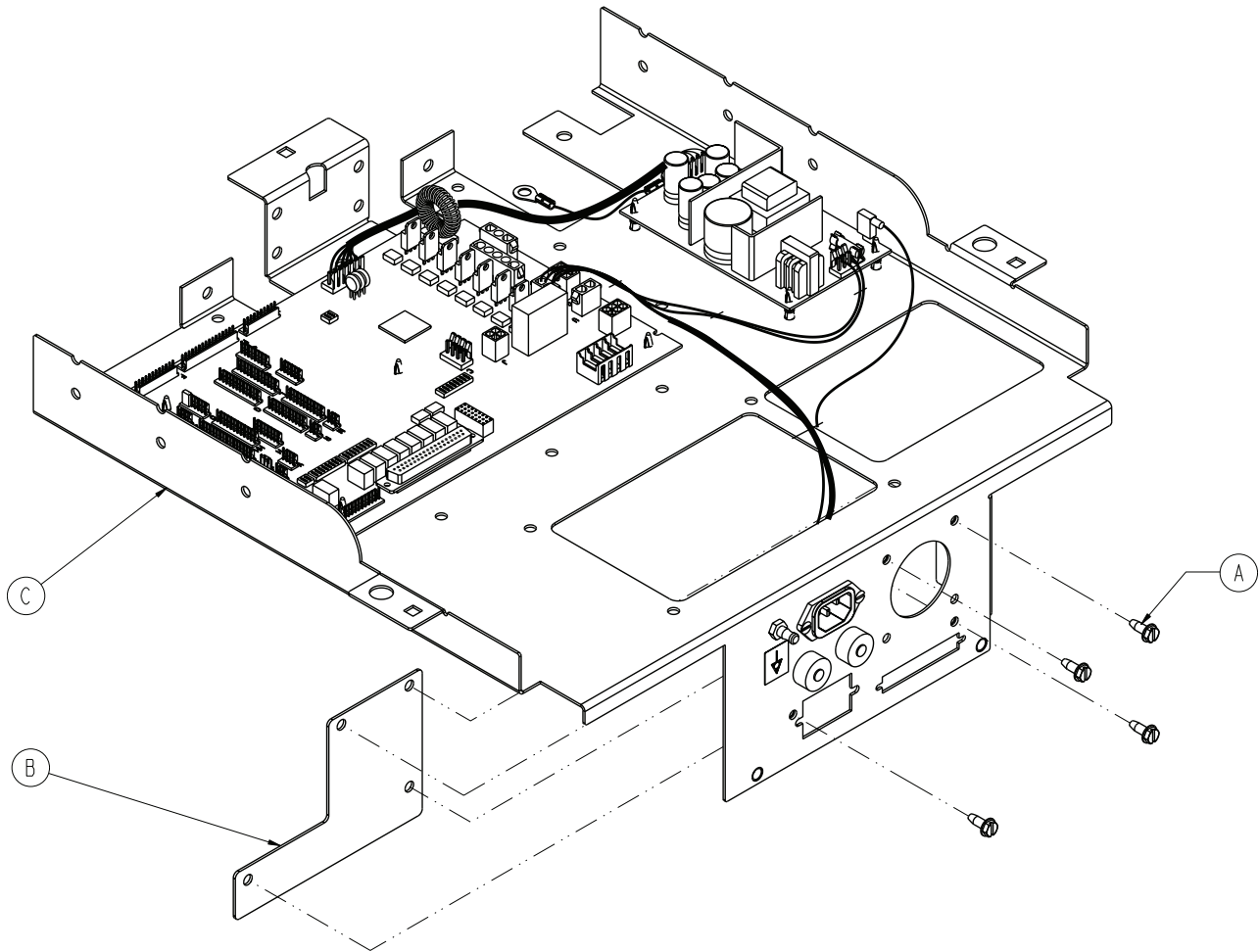

A	0003-224-000	4
B	4701-080-024	1
C	4701-080-030	1
D	4701-080-000	1

Rev A

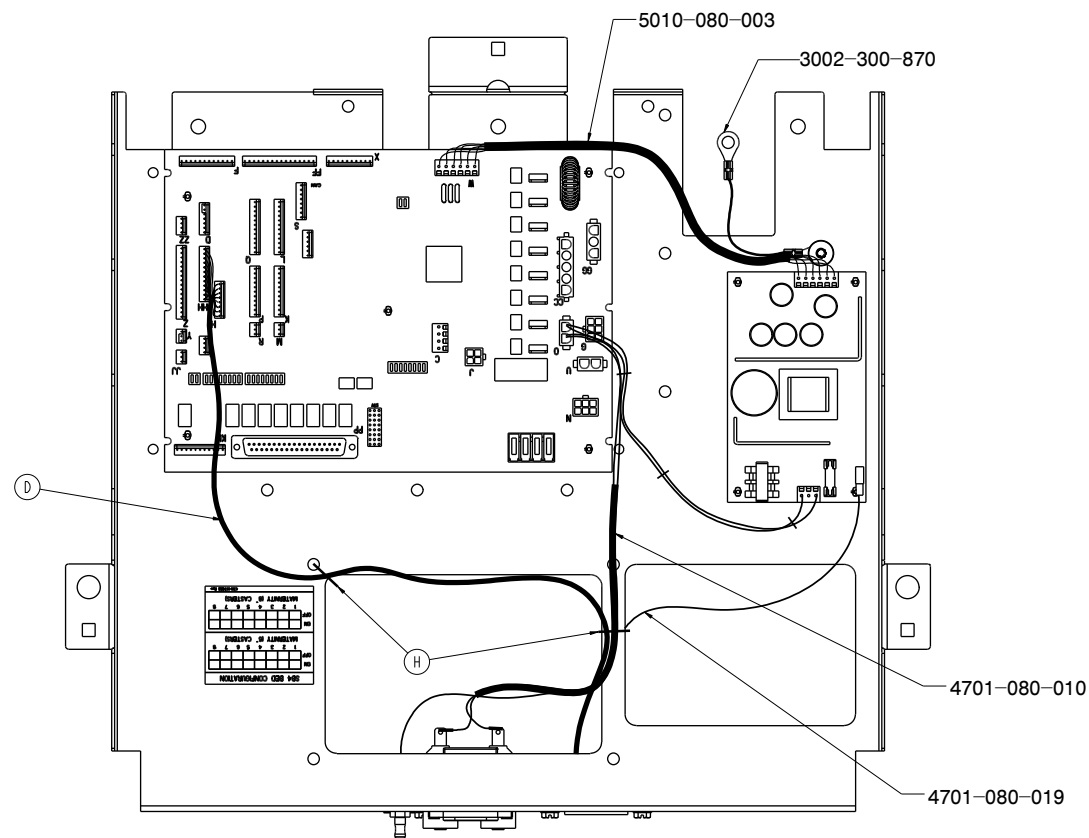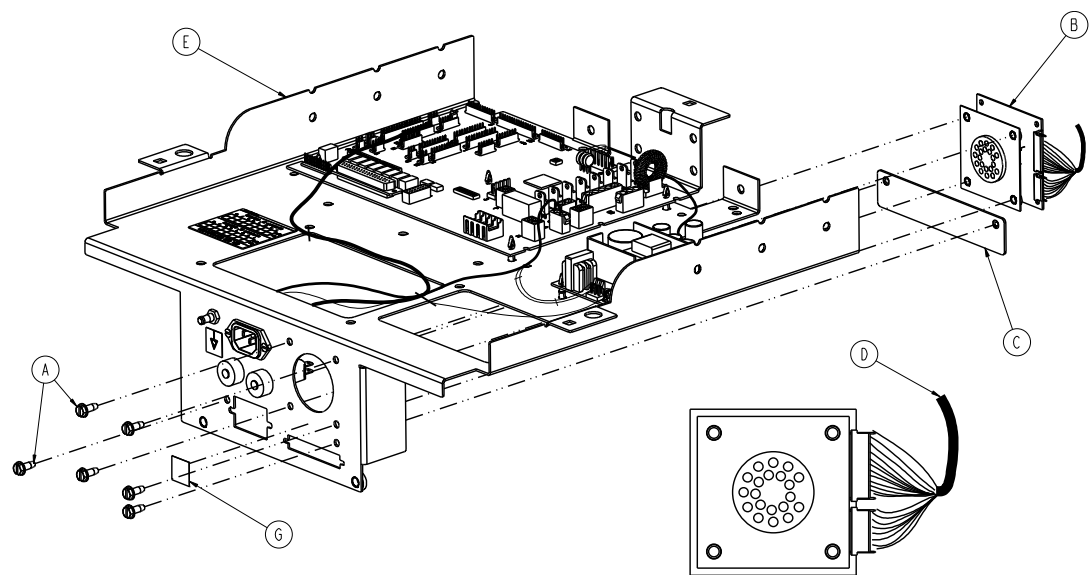


A	0003-224-000	6
B	3001-314-920	1
C	4701-080-022	1
D	4701-080-006	1

4701-180-040


E	4701-080-030	1
F	4701-080-000	1
G	4701-090-057	1
H	0059-086-000	2

Rev A

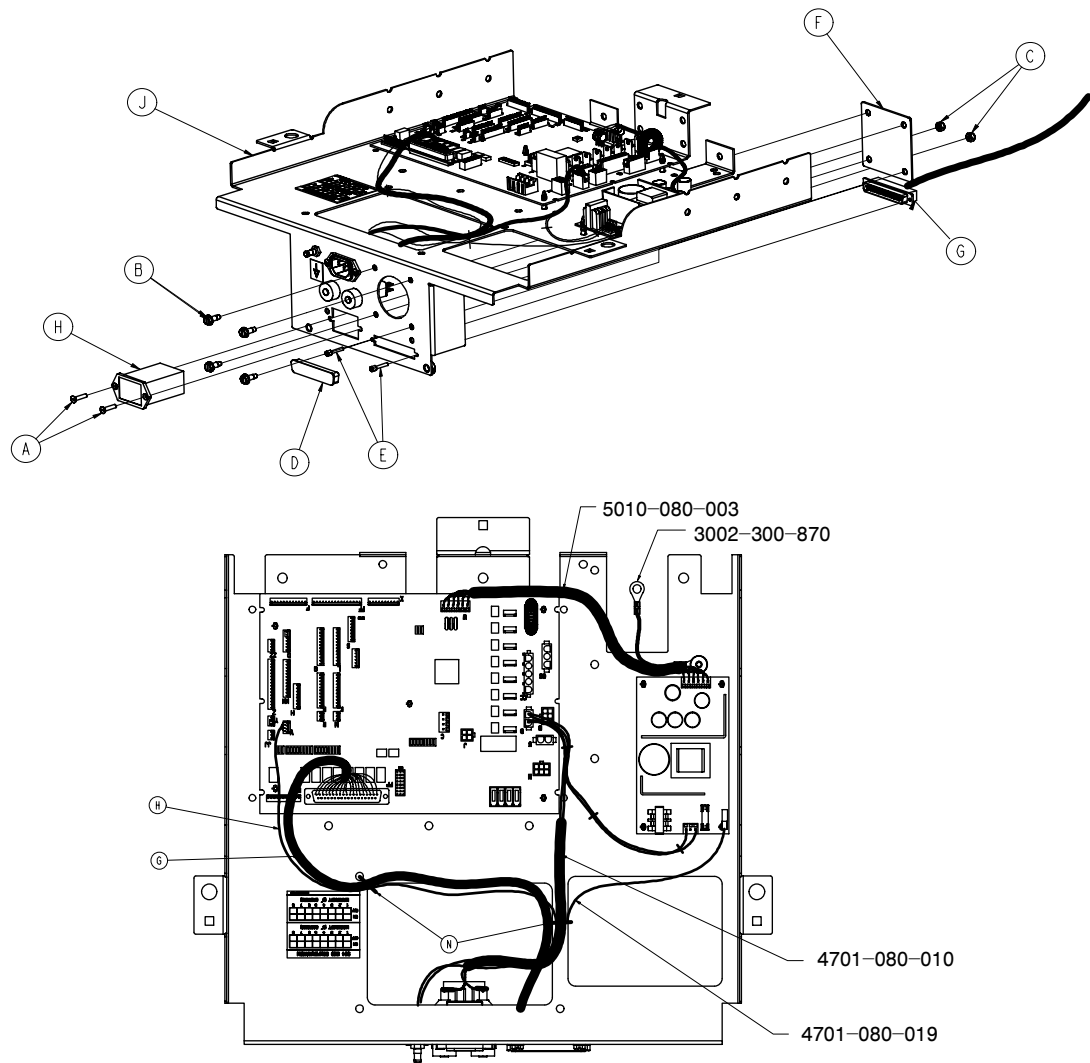

A	0001-087-000	2
B	0003-224-000	4
C	0016-023-000	2
D	0059-710-000	1
E	3001-300-007	2
F	4701-080-023	1
G	4701-080-009	1
H	5010-080-020	1
J	4701-080-030	1
K	4701-080-000	1

Rev A

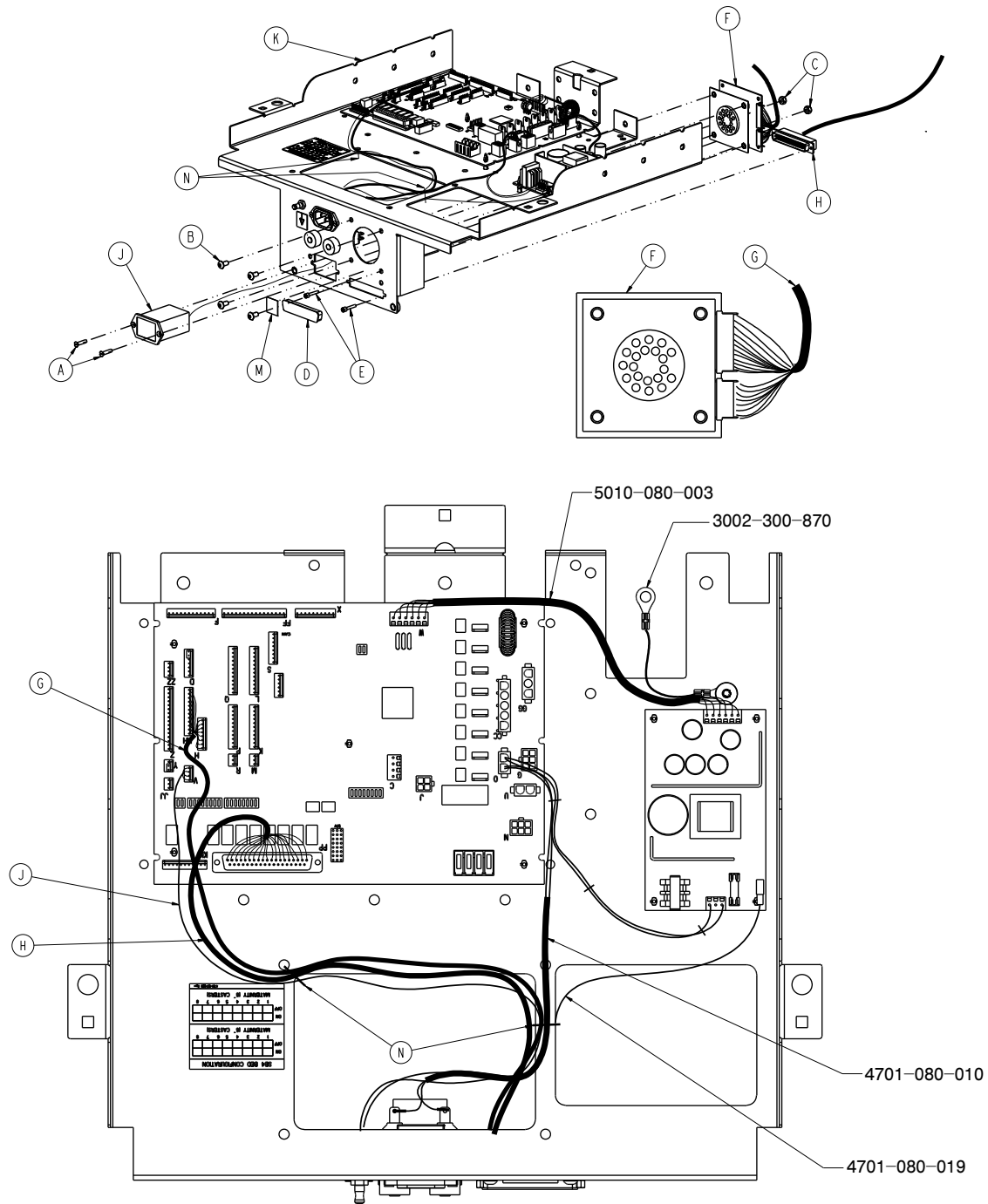



A	0001-087-000	2
B	0004-307-000	4
C	0016-023-000	2
D	0059-710-000	1
E	3001-300-007	2
F	3001-314-920	1

4701-108-055

G	4701-080-006	1
H	4701-080-009	1
J	5010-080-020	1
K	4701-080-030	1
L	4701-080-000	1
M	4701-090-057	1
N	0059-086-000	2

Rev E

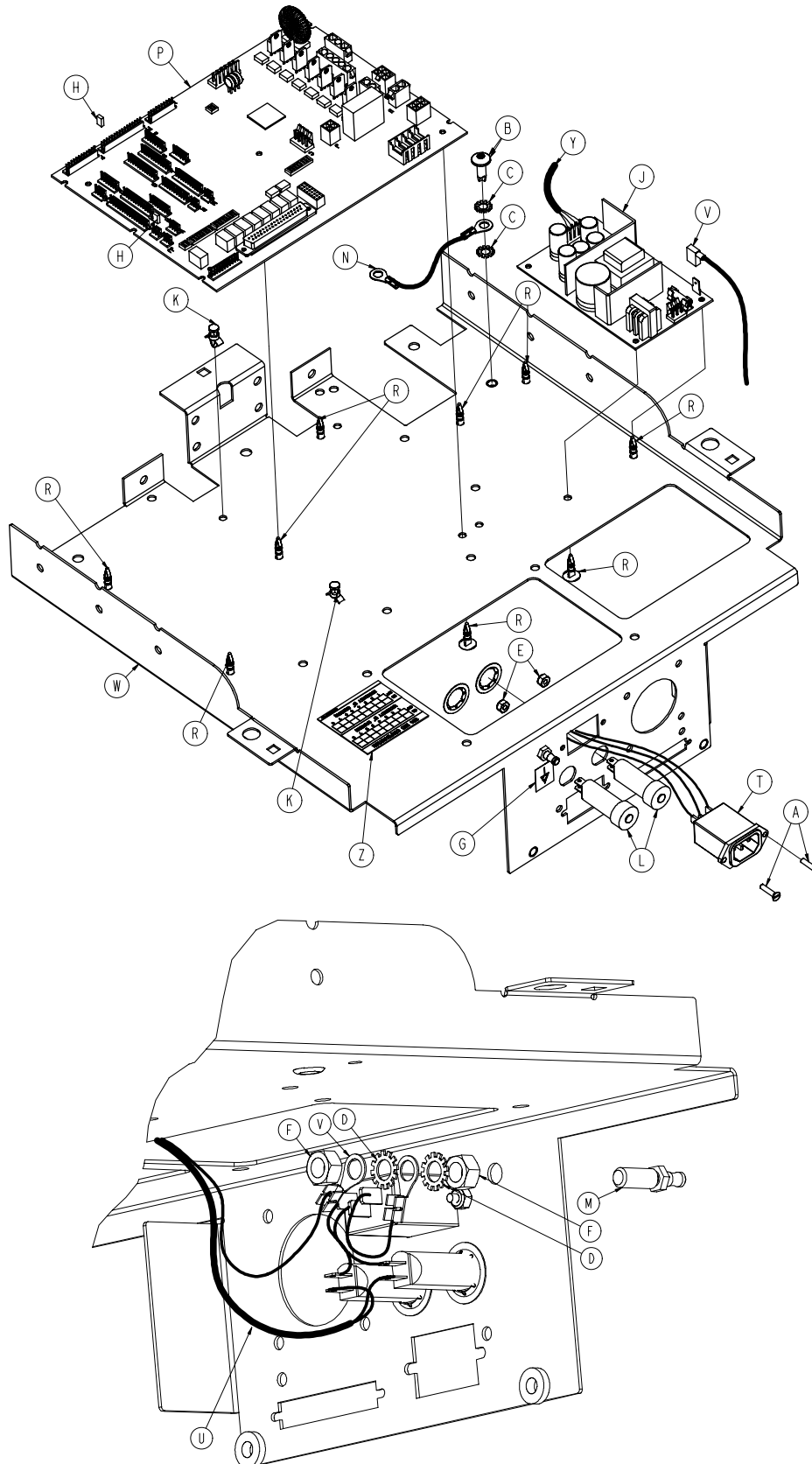

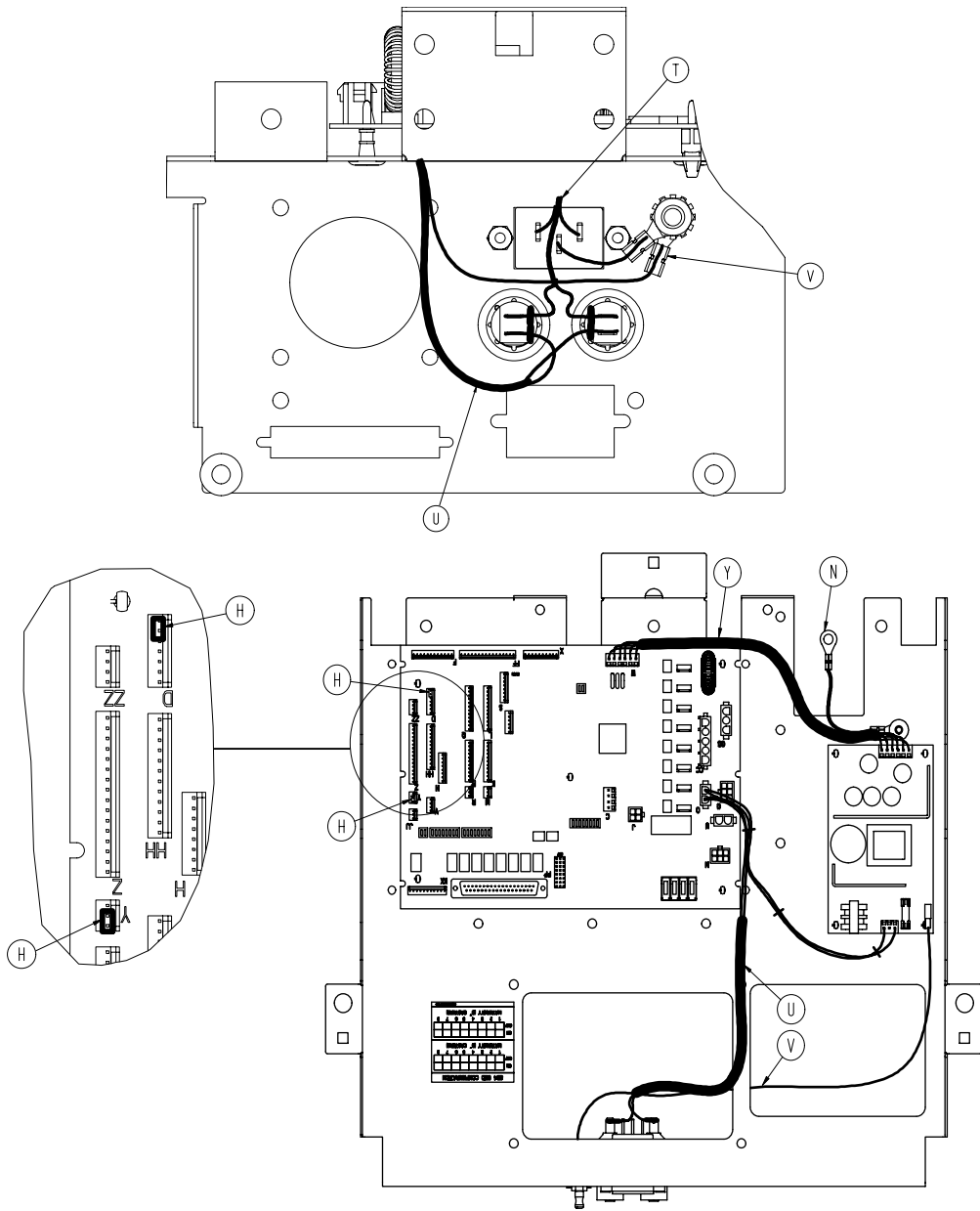


A	0001-087-000	2
B	0007-065-000	1
C	0013-010-000	2
D	0013-038-000	2
E	0016-023-000	2
F	0016-033-000	2
G	0036-115-000	1
H	0059-137-000	2
J	0059-157-000	1


4701-080-030

K	0059-171-000	2
L	0059-196-000	2
M	2011-001-215	1
N	3002-300-870	1
P	3002-407-950	1
R	3000-300-115	9
T	4701-080-004	1
U	4701-080-010	1
V	4701-080-019	1
W	4701-080-020	1
Y	5010-080-003	1
Z	4701-407-950	1


Rev F

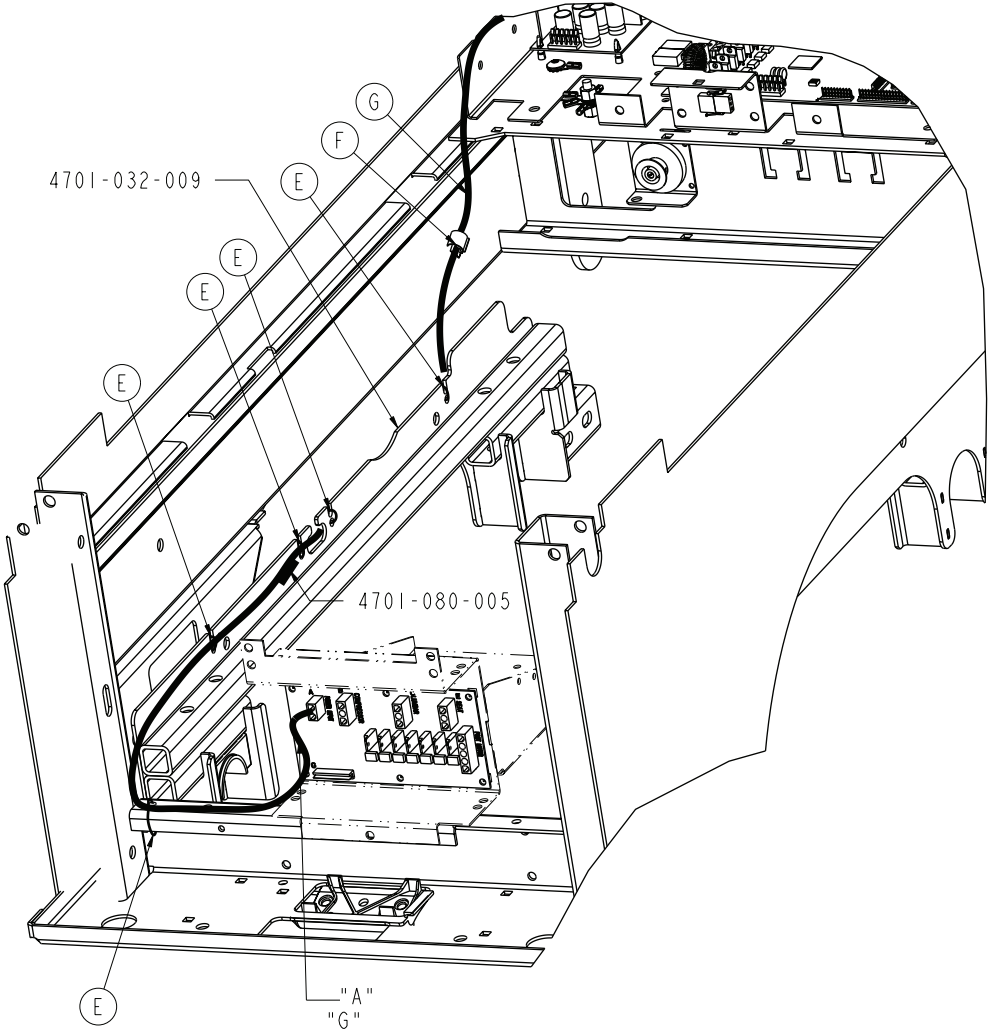



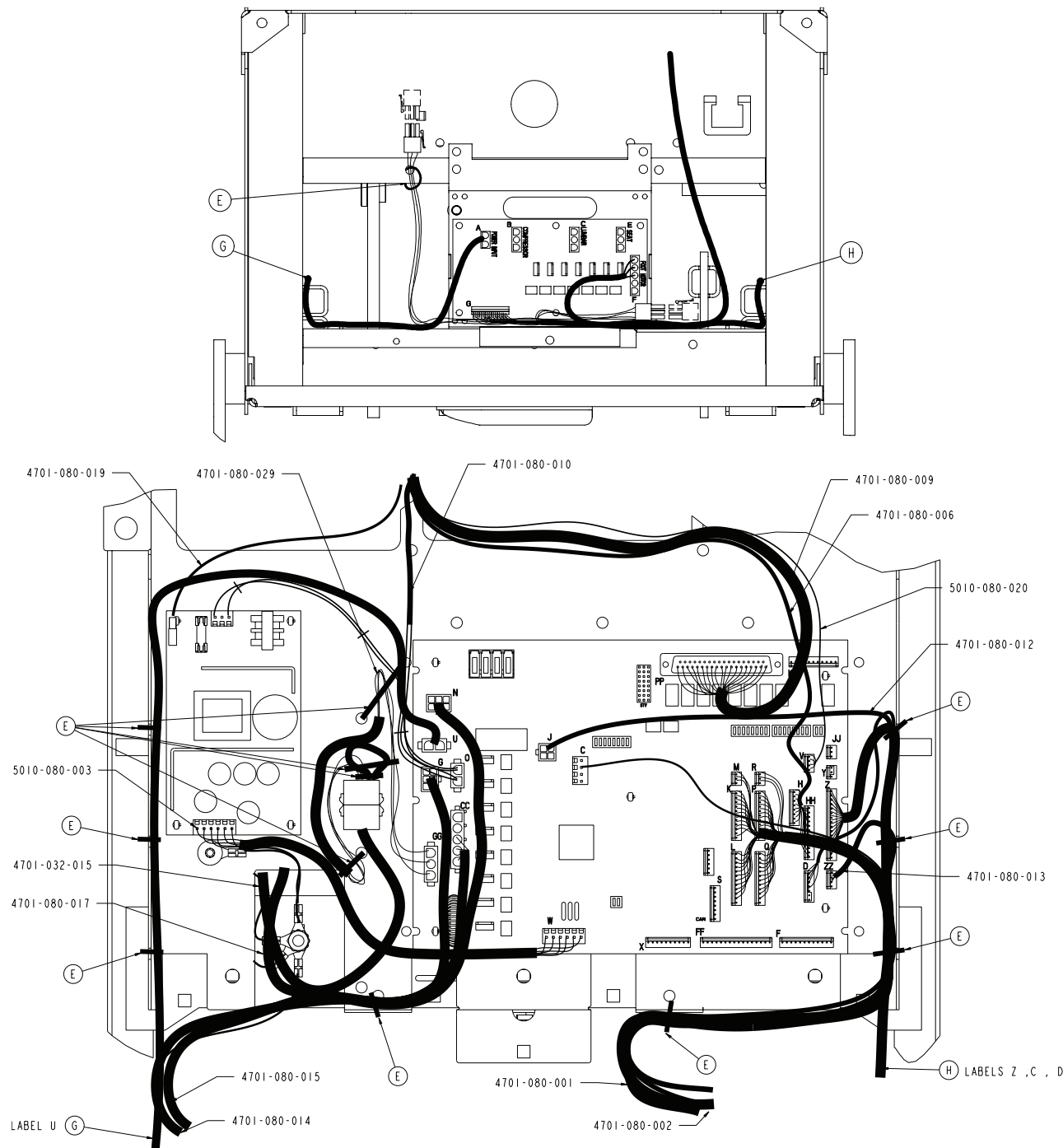


4701-080-000

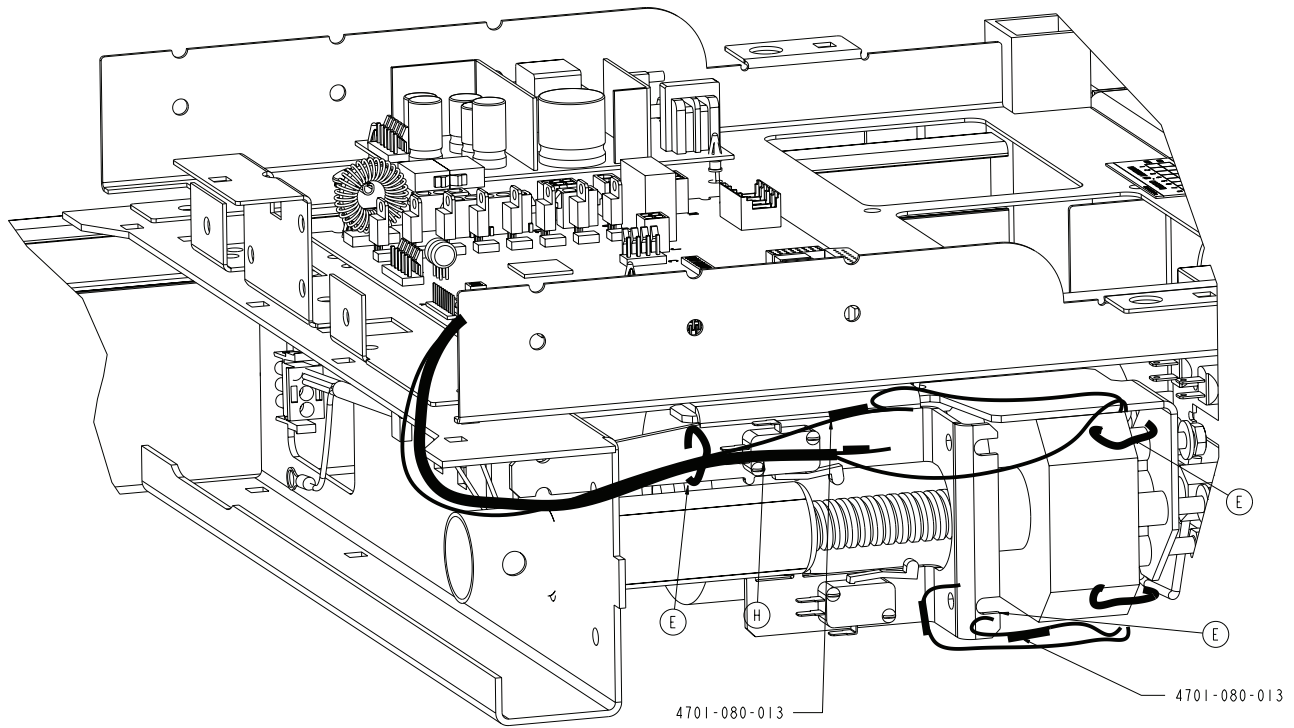


A	0005-030-000	1
B	0011-003-000	1
C	0013-010-000	4
D	0016-006-000	2
E	0059-086-000	31
F	8815-001-100	2
G	4701-080-005	1
H	4701-080-801	1
J	4701-080-012	1
K	4701-080-017	1



Rev A

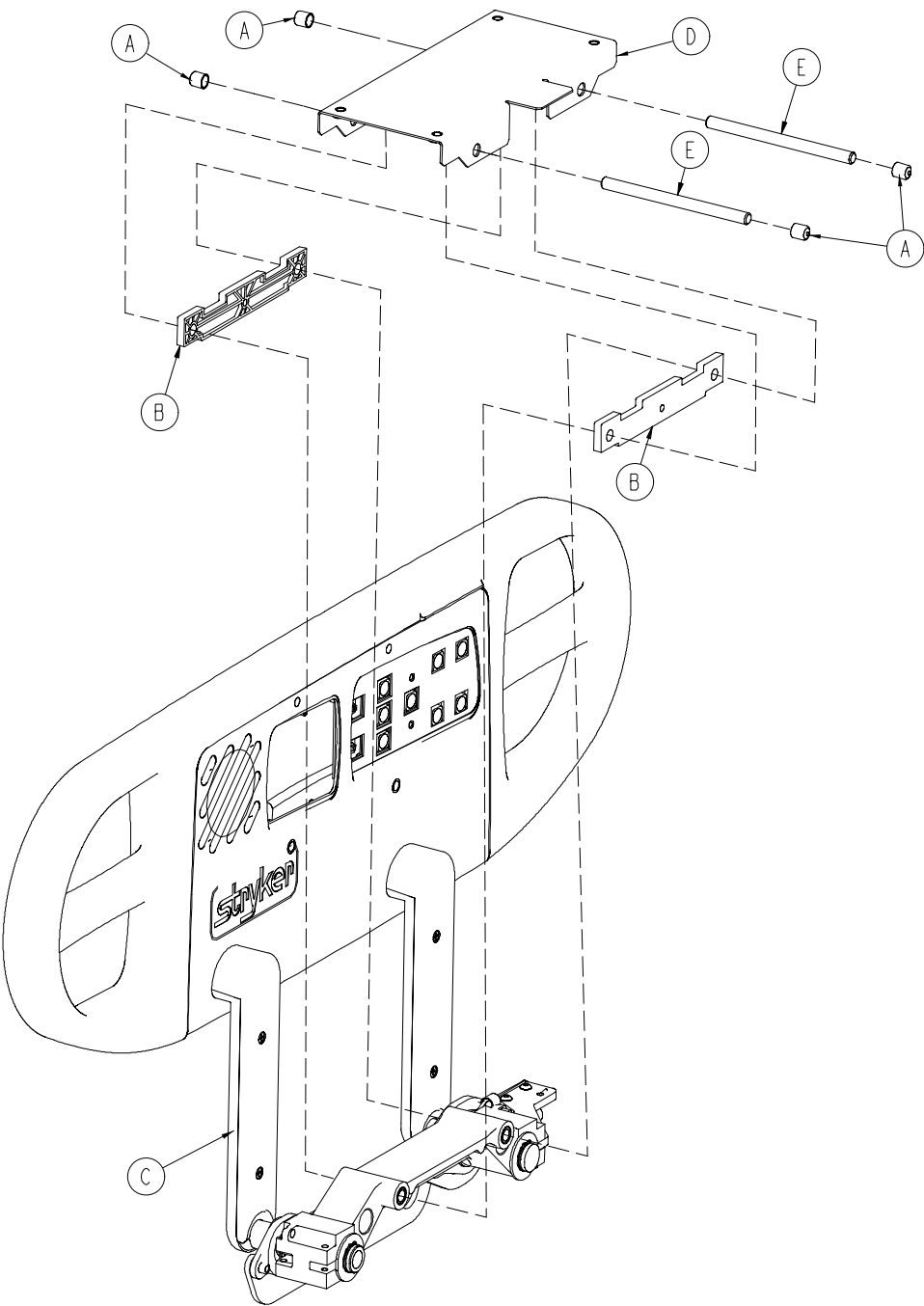


A	0056-021-000	4
B	3002-400-511	2
C	4701-120-085	1
D	4701-030-155	1
E	4701-030-156	2

Rev A

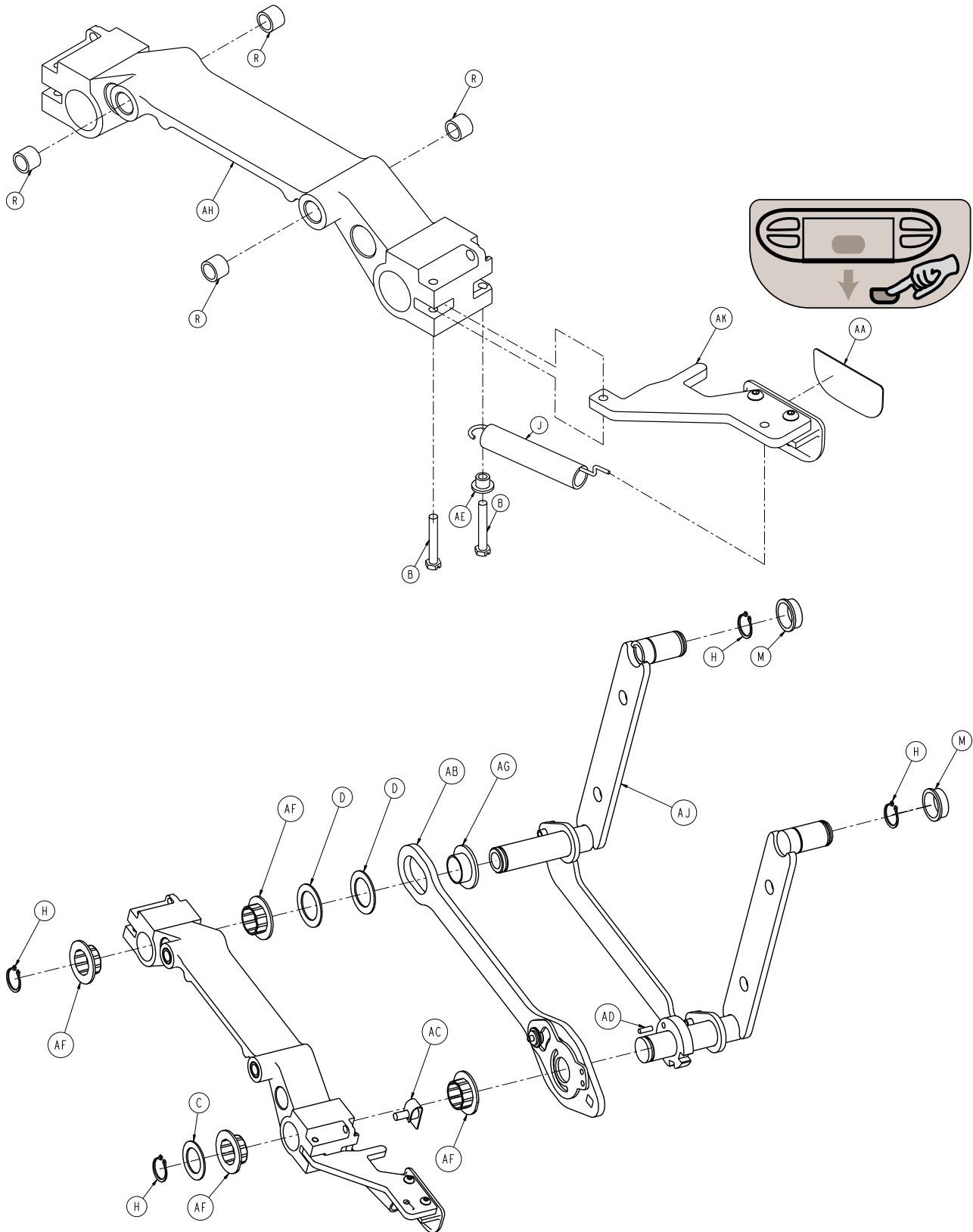

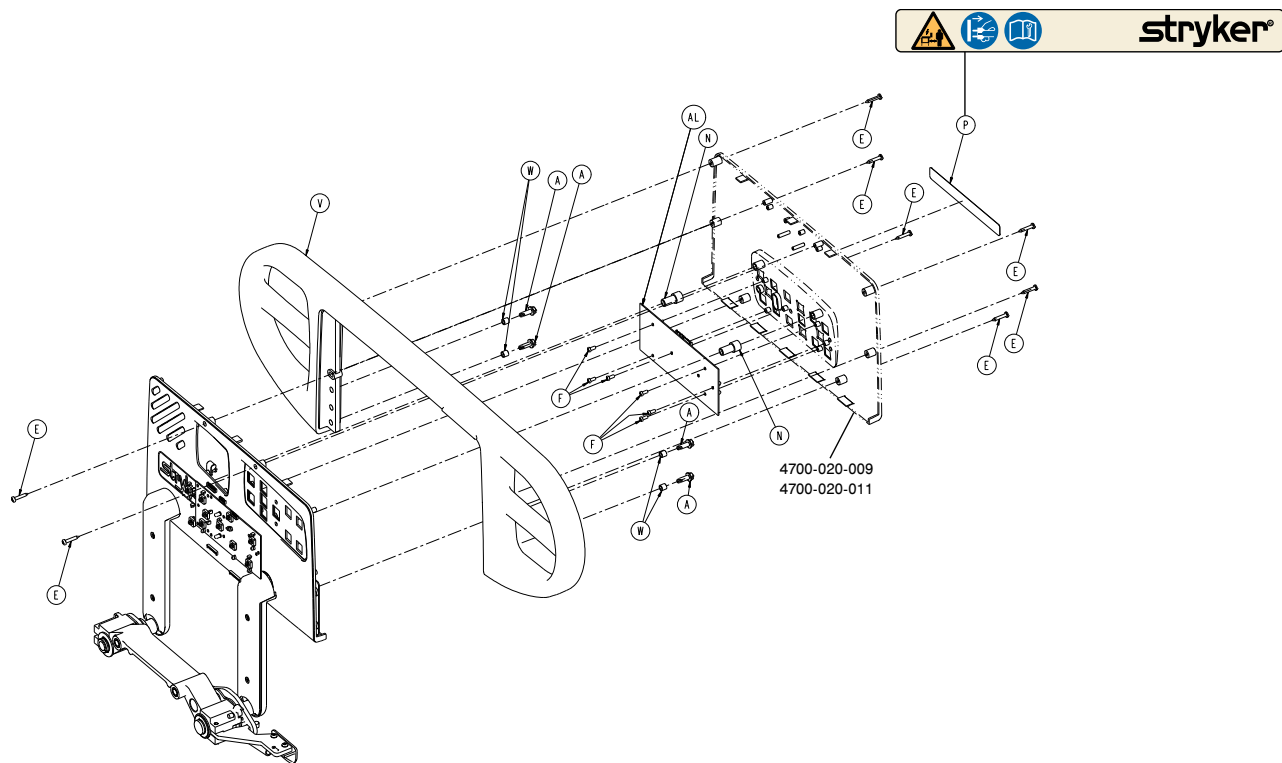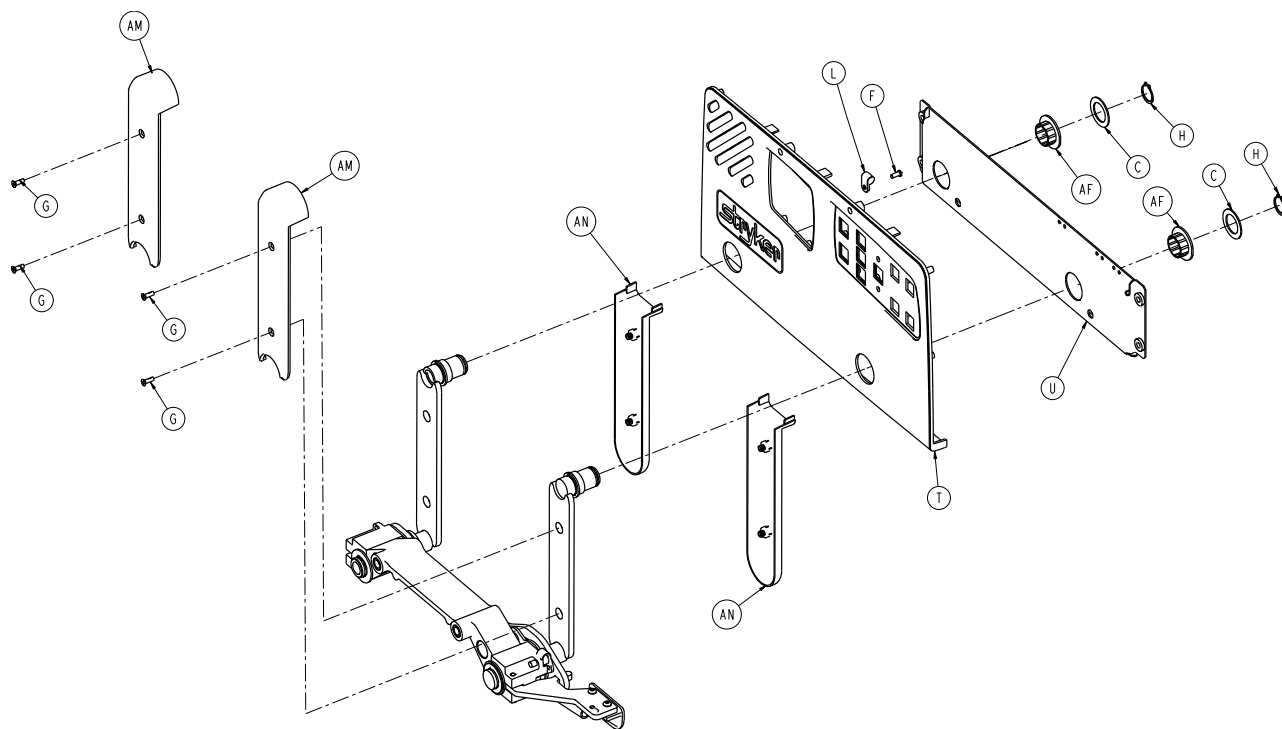


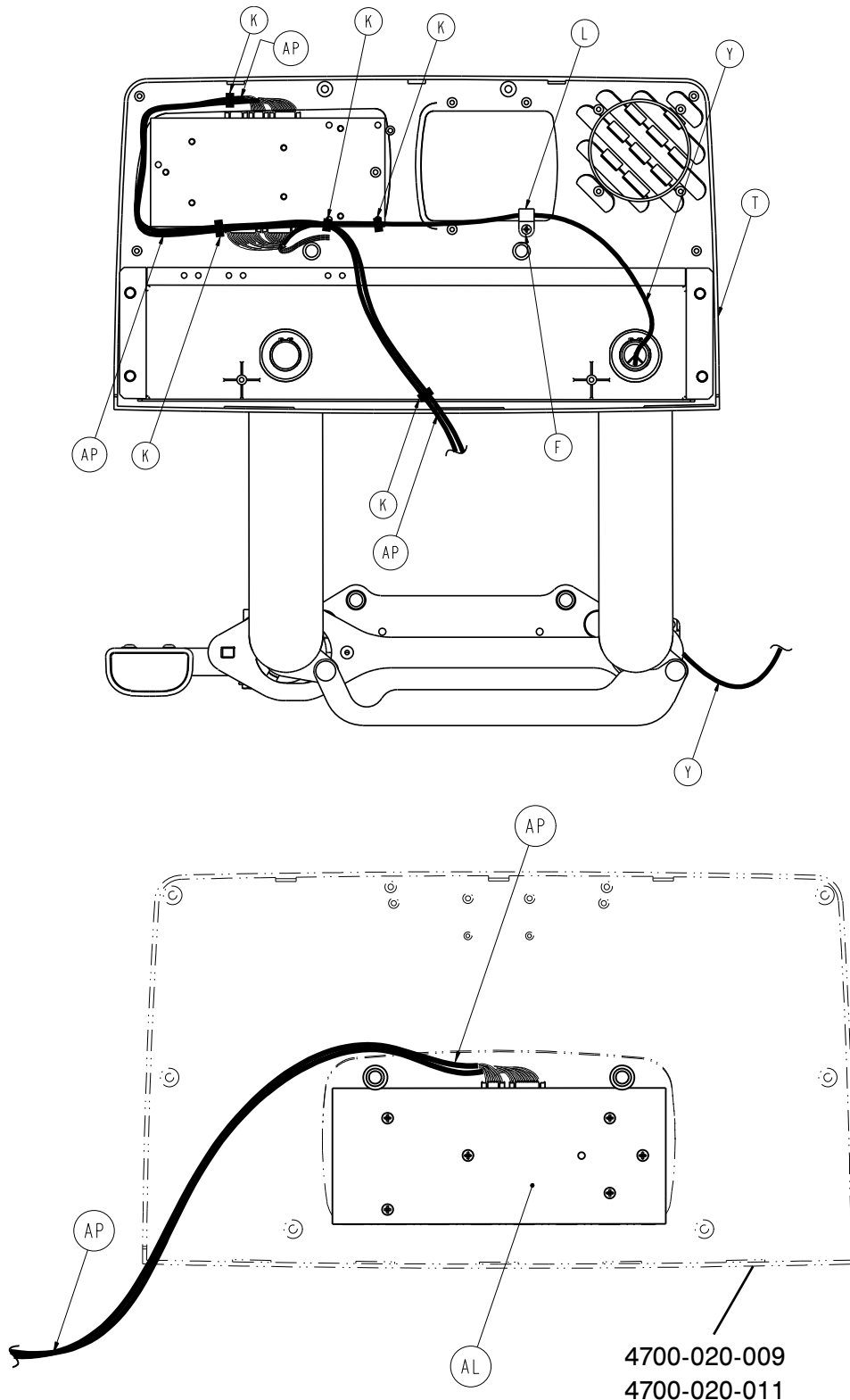

4701-120-085


A	0003-226-000	4
B	0003-344-000	2
C	0011-343-000	3
D	0014-093-000	2
E	0023-090-000	8
F	0023-088-000	7
G	0001-072-000	4
H	0028-128-000	6
J	3000-200-334	1
K	3000-300-114	5
L	3000-300-478	1
M	3000-400-513	2
N	3000-400-523	2
P	4701-109-974	1
R	3000-400-557	4
T	3001-400-040	1
U	3001-400-130	1
V	3001-400-515	1
W	3001-400-558	4
Y	4701-080-026	1
AA	4701-109-971	1
AB	3002-400-070	1
AC	3002-400-090	1
AD	3002-400-505	1
AE	3002-400-509	1
AF	3002-400-513	6
AG	3002-400-519	1
AH	3003-400-528	1
AJ	4700-320-005	1
AK	4700-220-007	1
AL	4701-080-070	1
AM	5000-020-005	2
AN	3001-400-619	2
AP	5000-080-012	1




Rev A

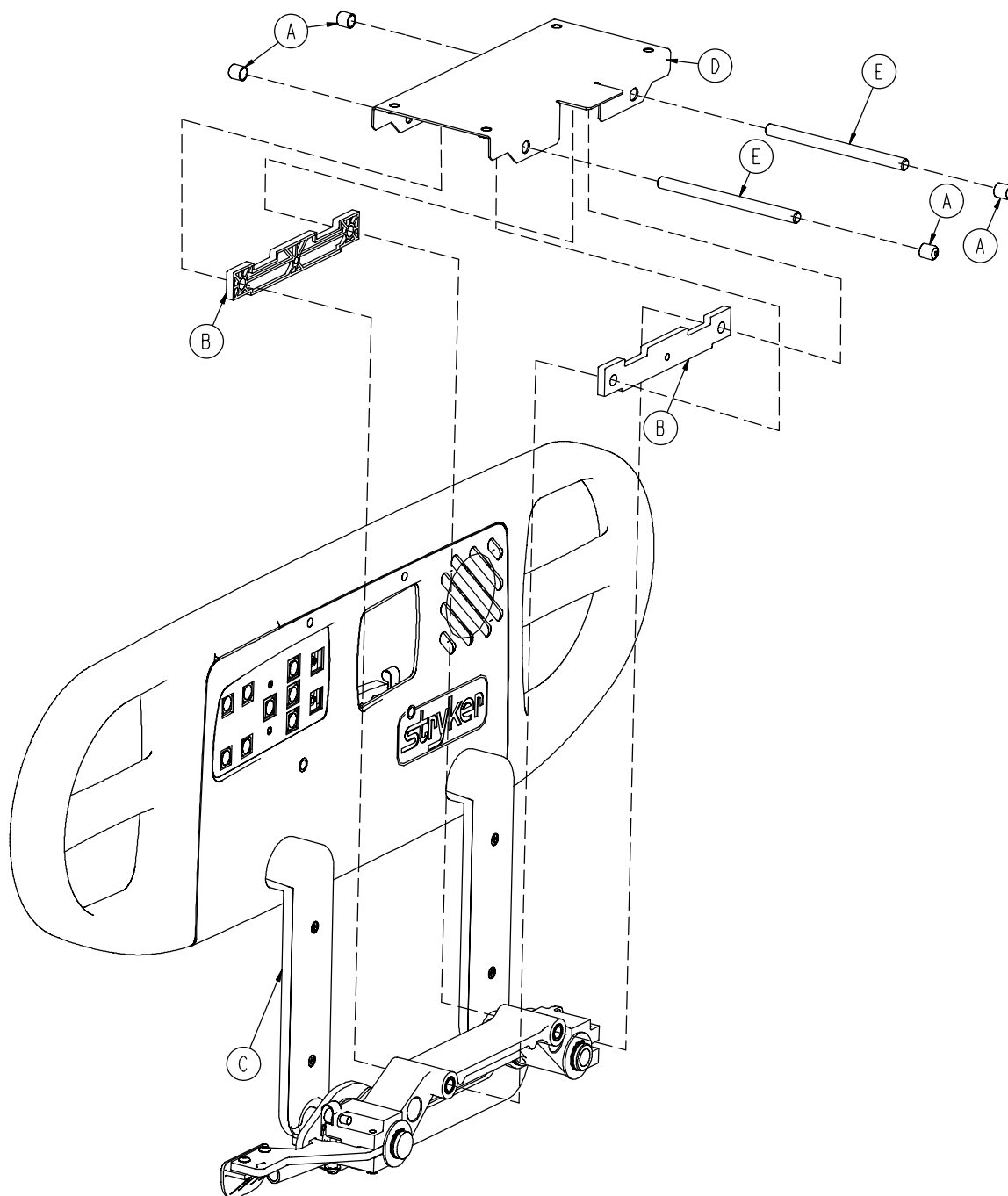


A	0056-021-000	4
B	3002-400-511	2
C	4701-120-090	1
D	4701-030-155	1
E	4701-030-156	2

Rev A

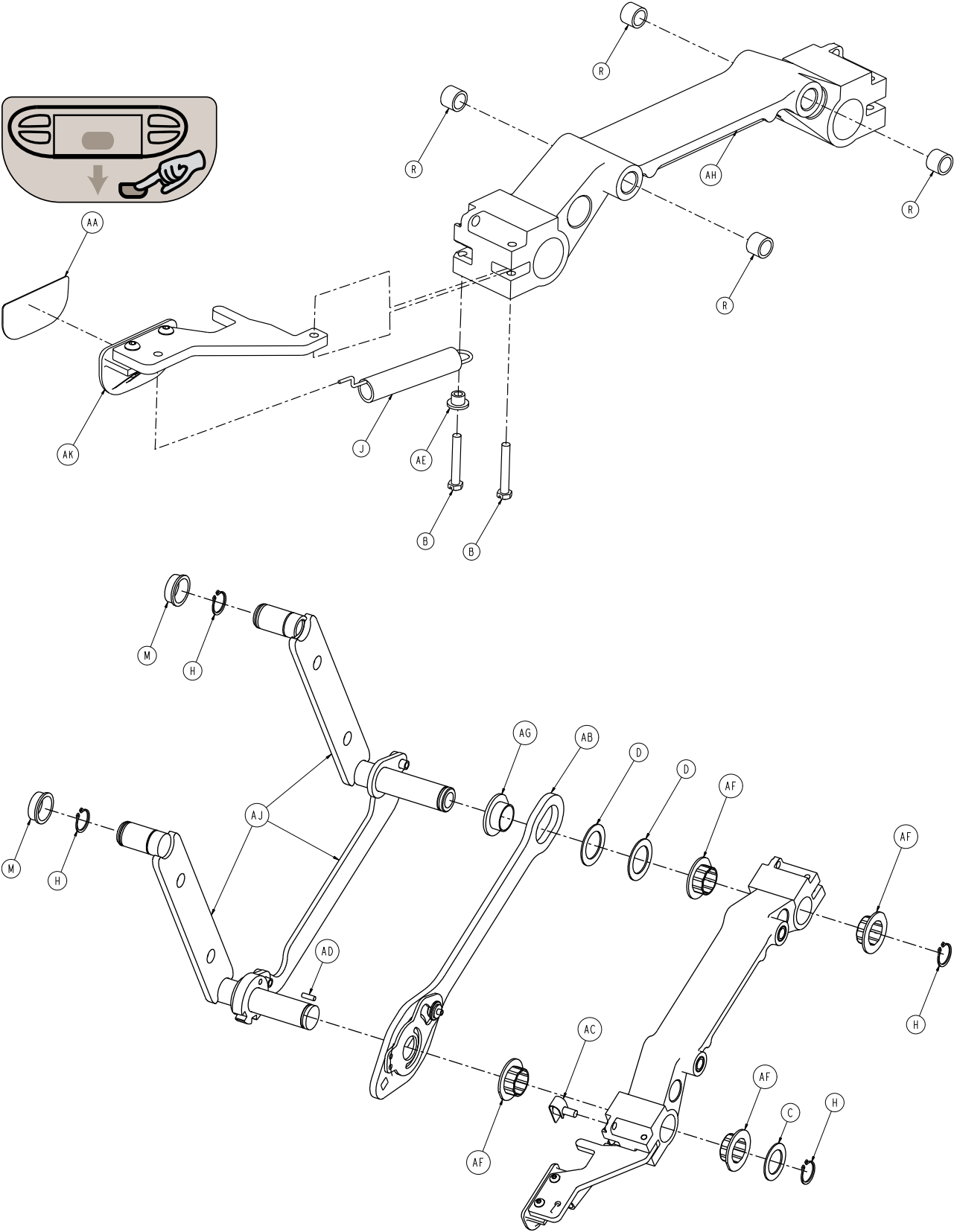

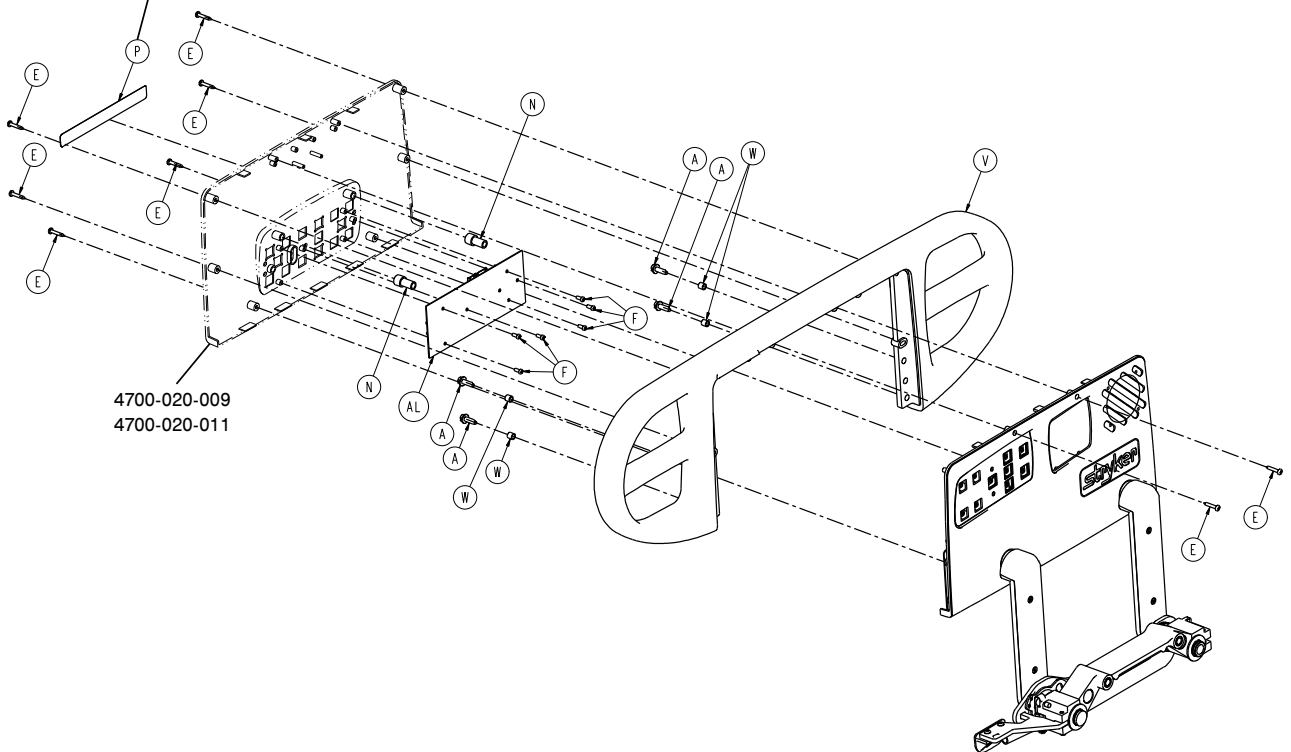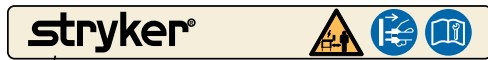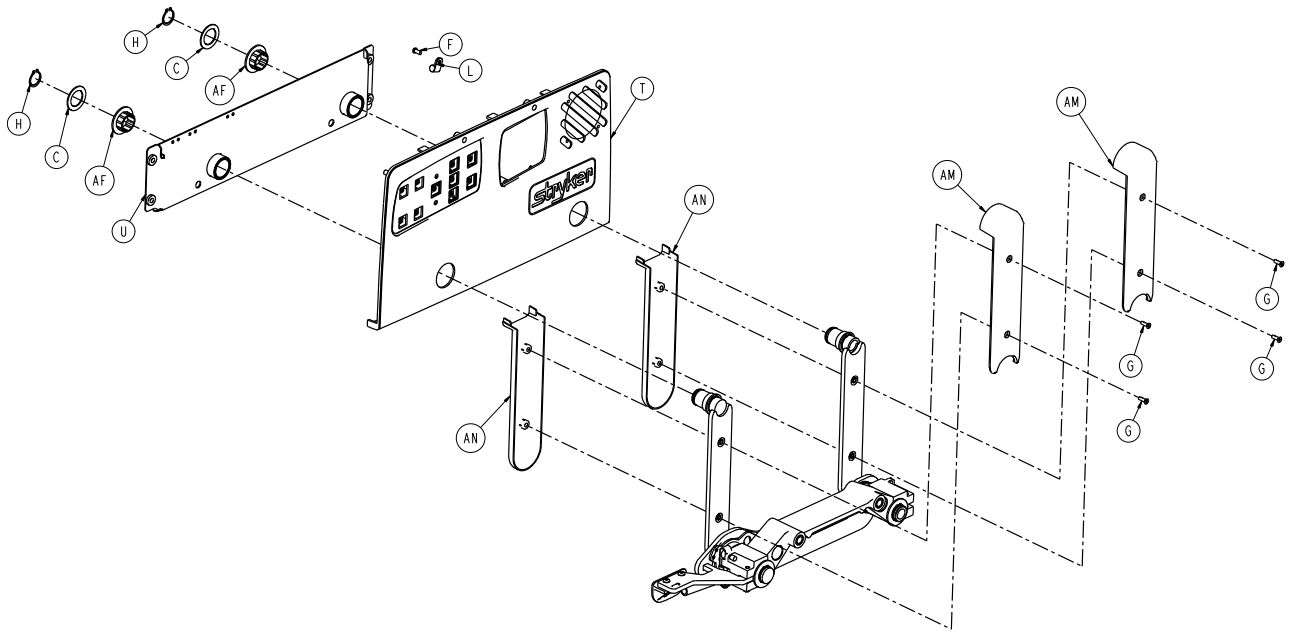


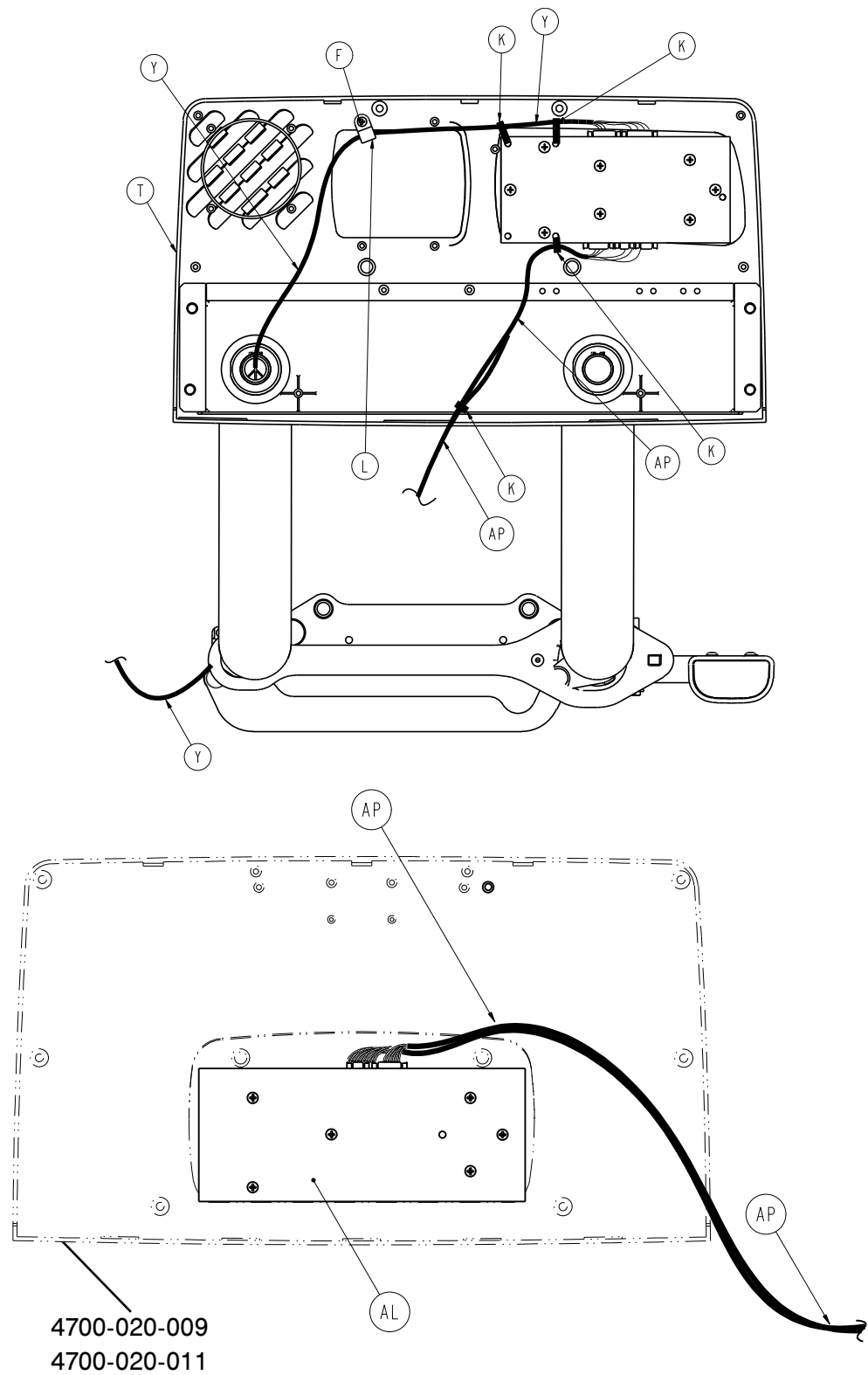

4701-120-090


A	0003-226-000	4
B	0003-344-000	2
C	0011-343-000	2
D	0014-093-000	2
E	0023-090-000	8
F	0023-088-000	7
G	0001-072-000	4
H	0028-128-000	6
J	3000-200-334	1
K	0059-086-000	5
L	3000-300-478	1
M	3000-400-513	2
N	3000-400-523	2
P	4701-109-973	1
R	3000-400-557	4
T	3001-400-045	1
U	3001-400-230	1
V	3001-400-515	1
W	3001-400-558	4
Y	4701-080-026	1
AA	4701-109-971	1
AB	3002-400-075	1
AC	3002-400-090	1
AD	3002-400-505	1
AE	3002-400-509	1
AF	3002-400-513	6
AG	3002-400-519	1
AH	3003-400-528	1
AJ	4700-320-004	1
AK	4700-220-006	1
AL	4701-080-060	1
AM	5000-020-005	2
AN	3001-400-619	2
AP	5000-080-011	1


Rev —

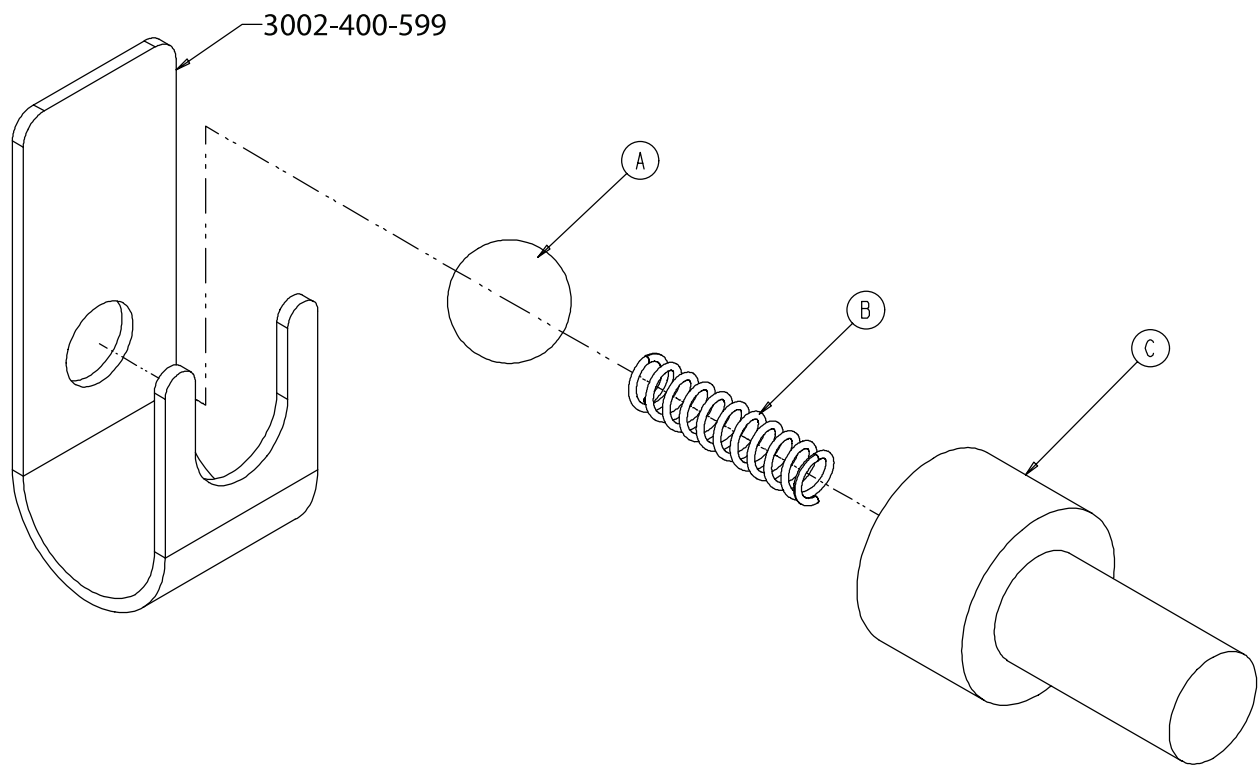


A	0031-137-000	1
B	0038-464-000	1
C	3002-400-524	1

Rev A

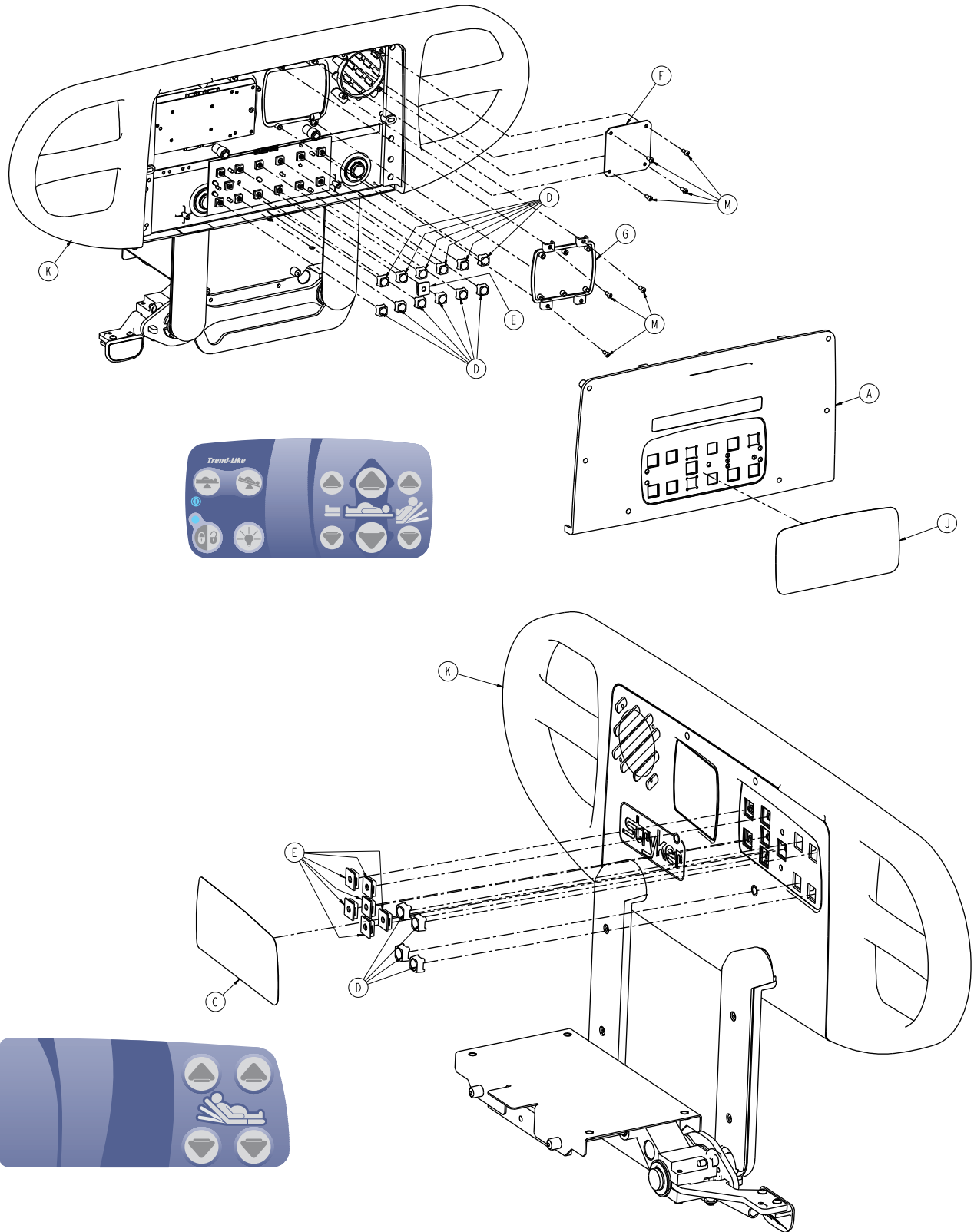


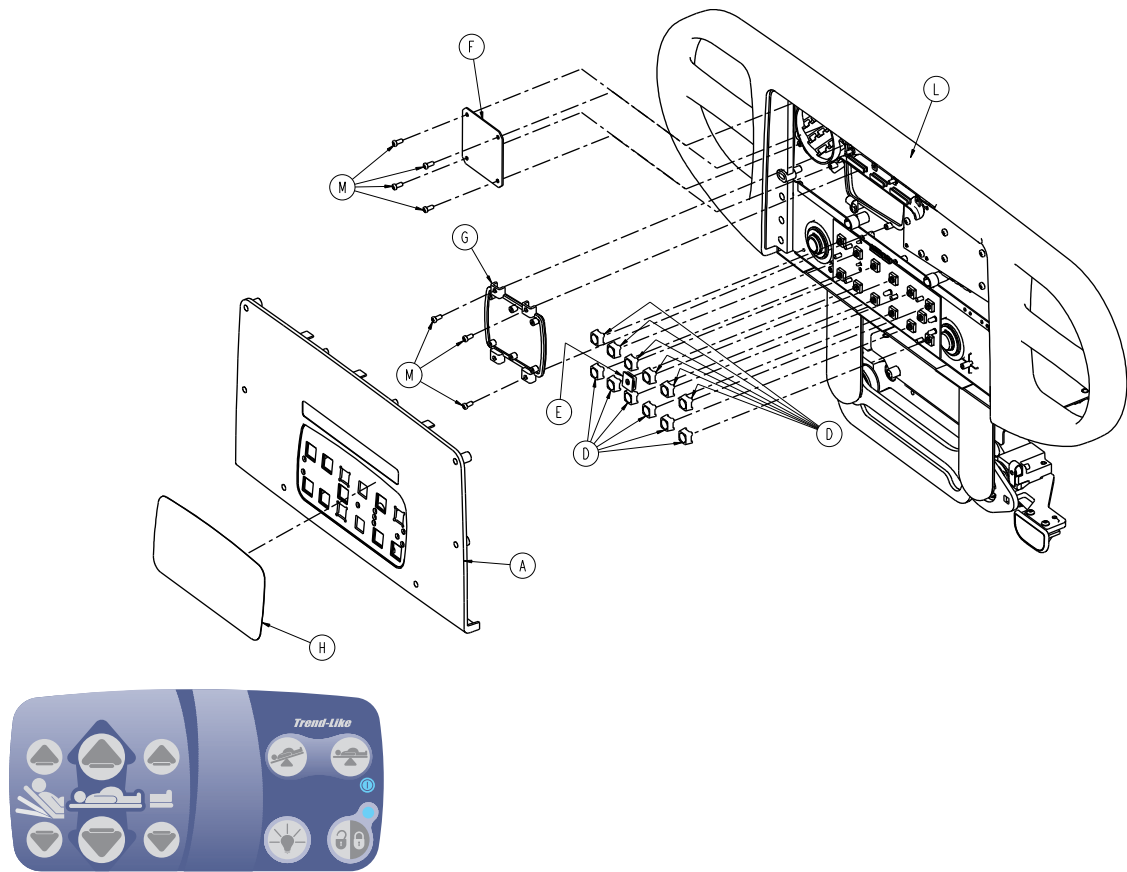


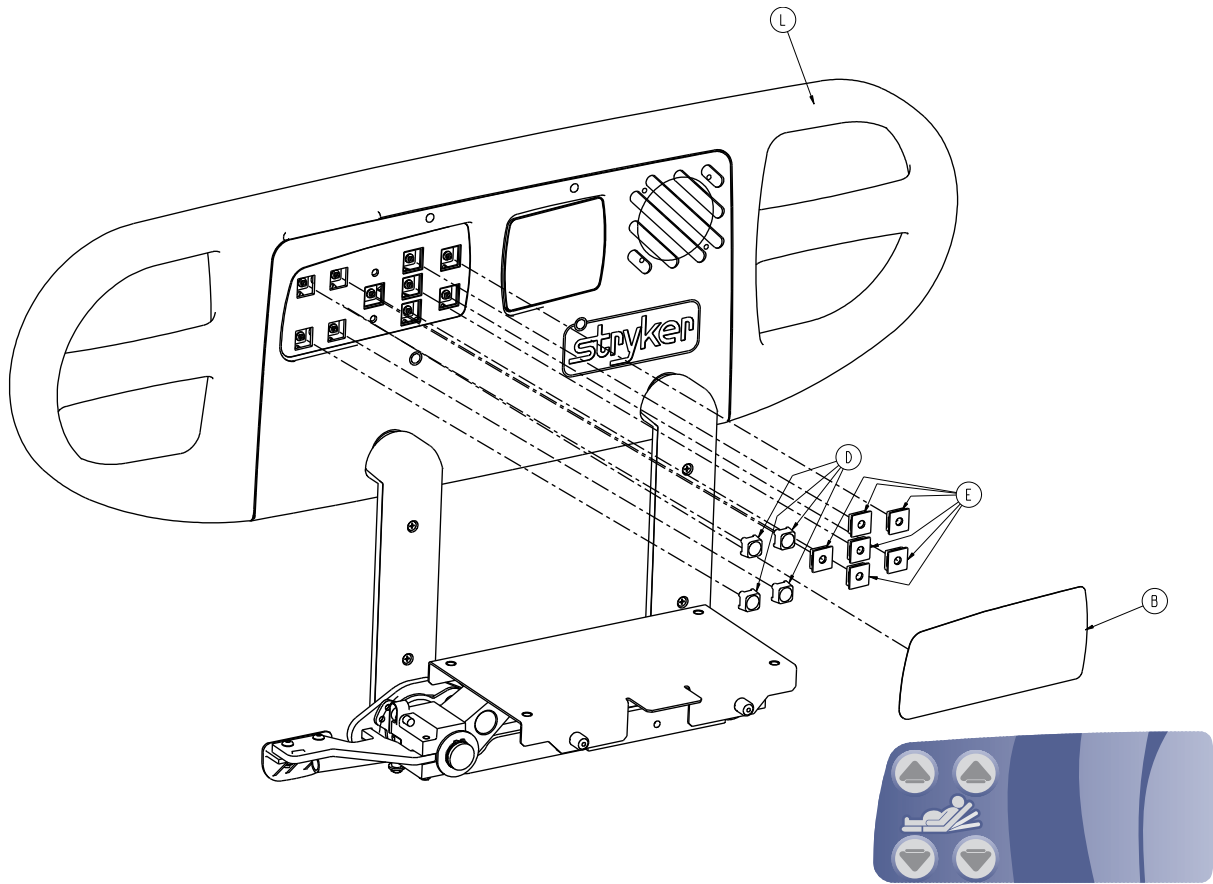


A	4700-020-011	2
B	4701-109-928	1
C	4701-109-929	1
D	3001-400-953	12
E	3001-400-522	14
F	3001-400-517	2
G	3001-400-535	2
H	4701-109-918	1
J	4701-109-919	1
K	4701-120-095	1
L	4701-120-100	1
M	0023-088-000	14

Rev A

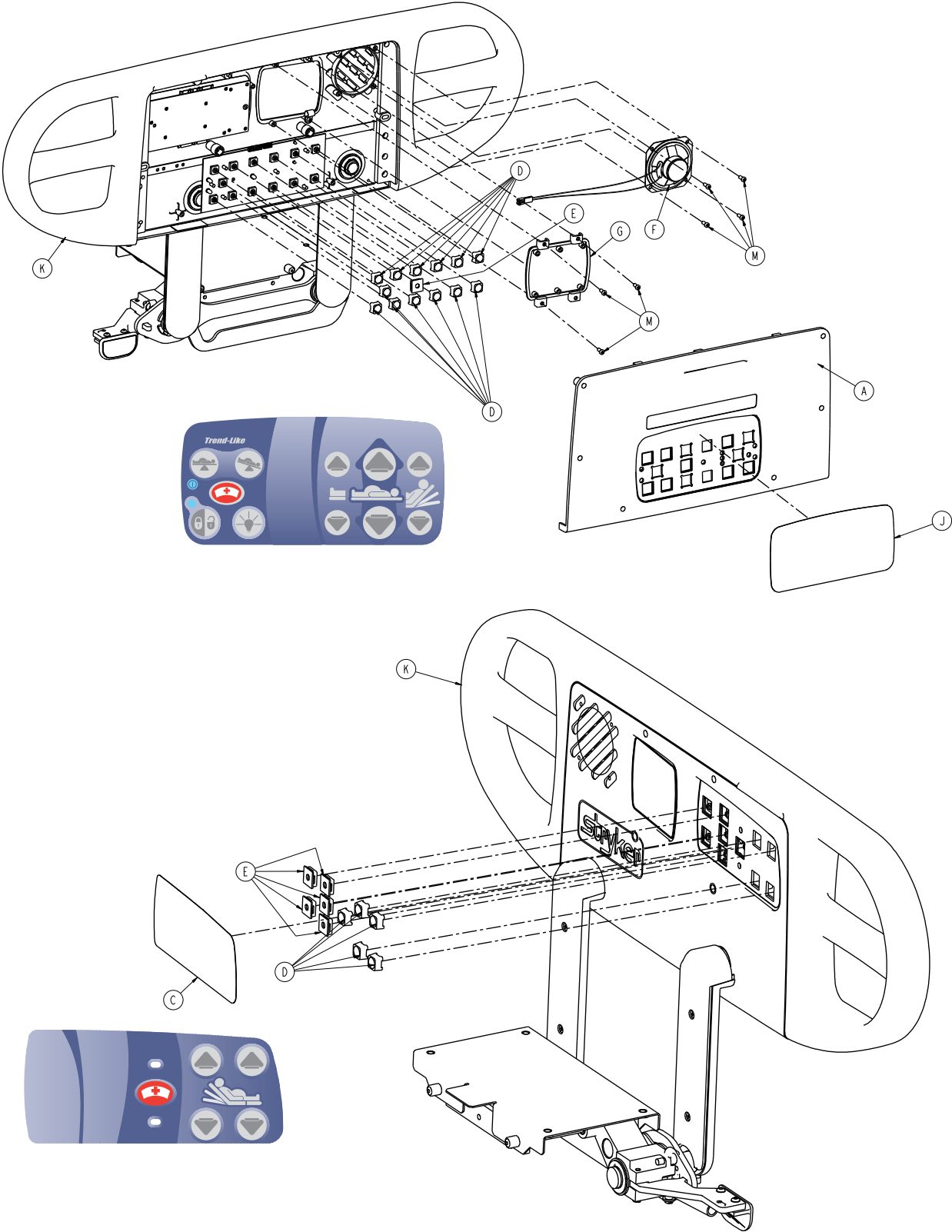


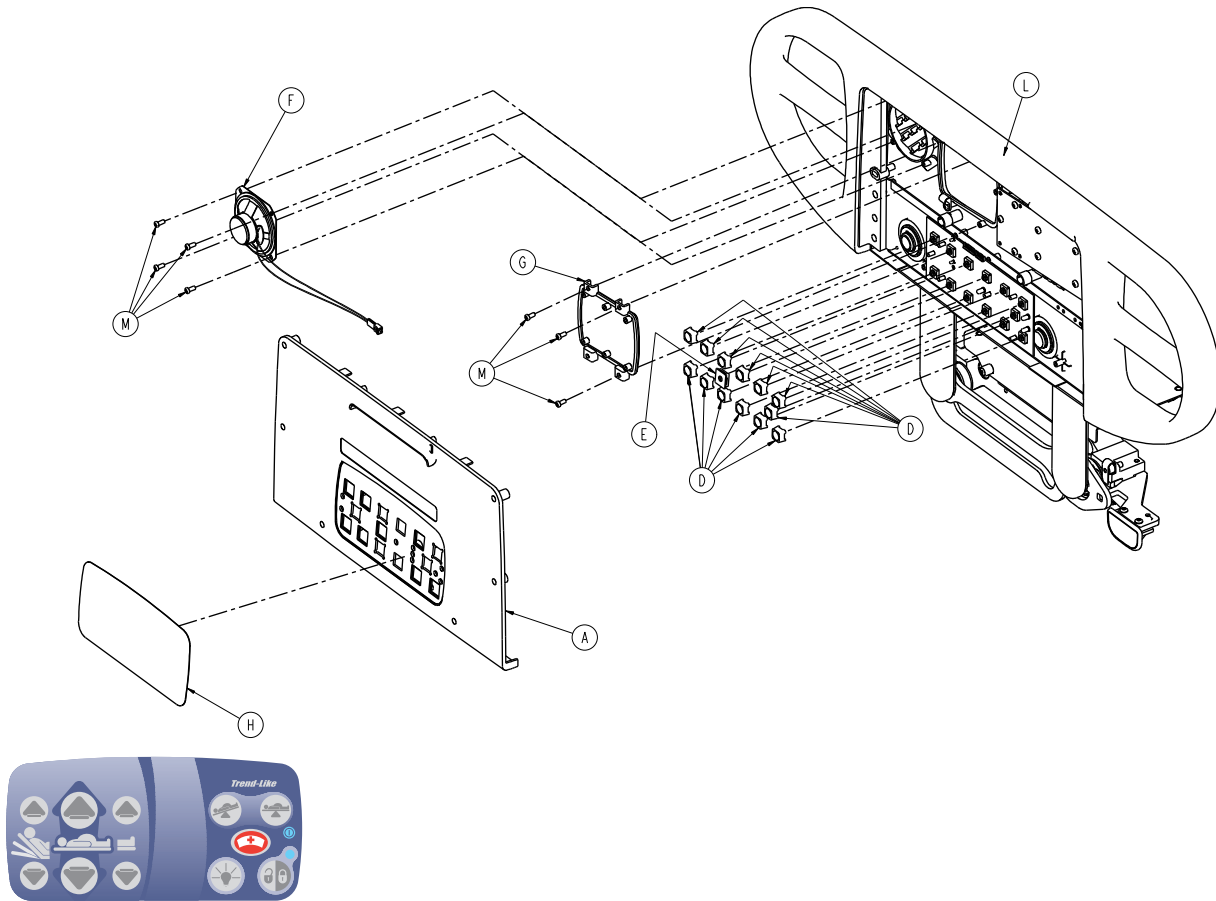


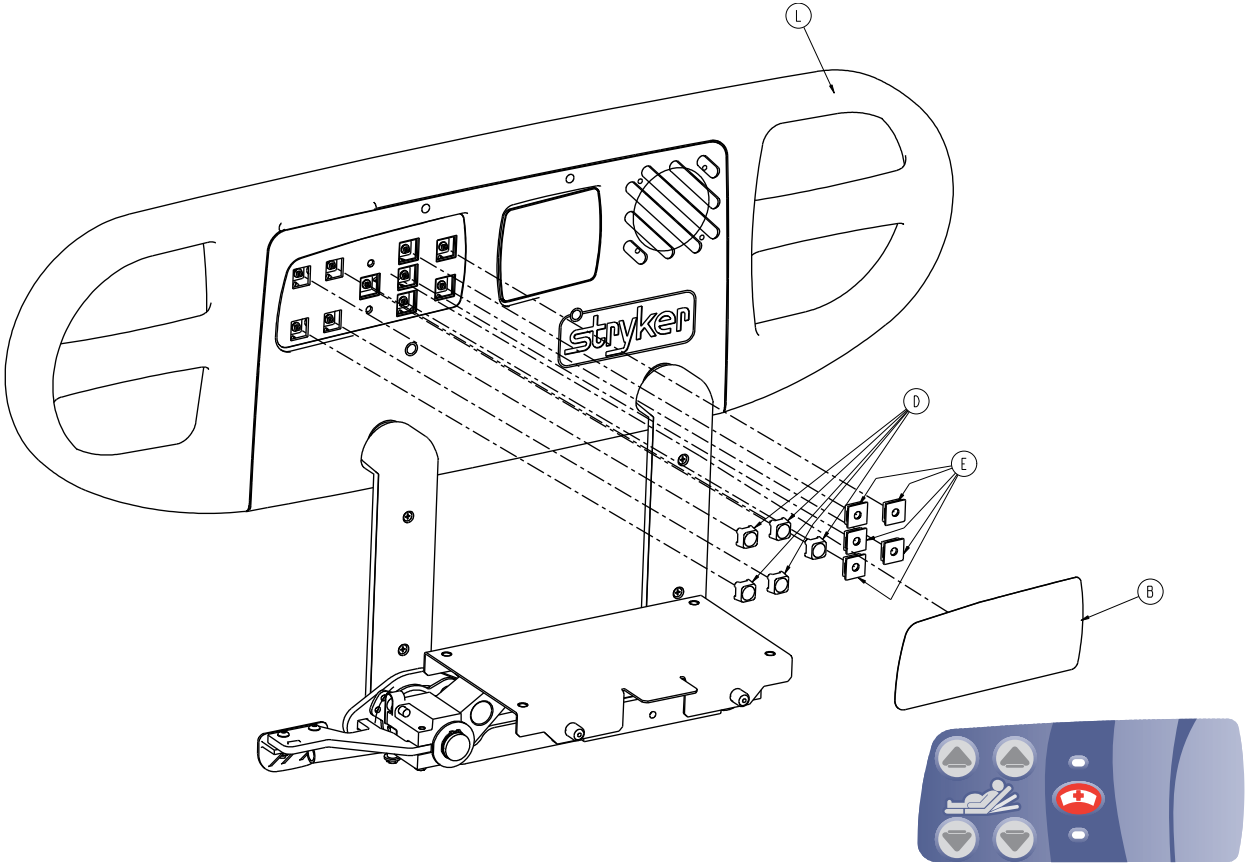


A	4700-020-009	2
B	4701-109-926	1
C	4701-109-927	1
D	3001-400-953	16
E	3001-400-522	12
F	3001-403-831	2
G	3001-400-535	2
H	4701-109-913	1
J	4701-109-914	1
K	4701-120-095	1
L	4701-120-100	1
M	0023-088-000	14

Rev A

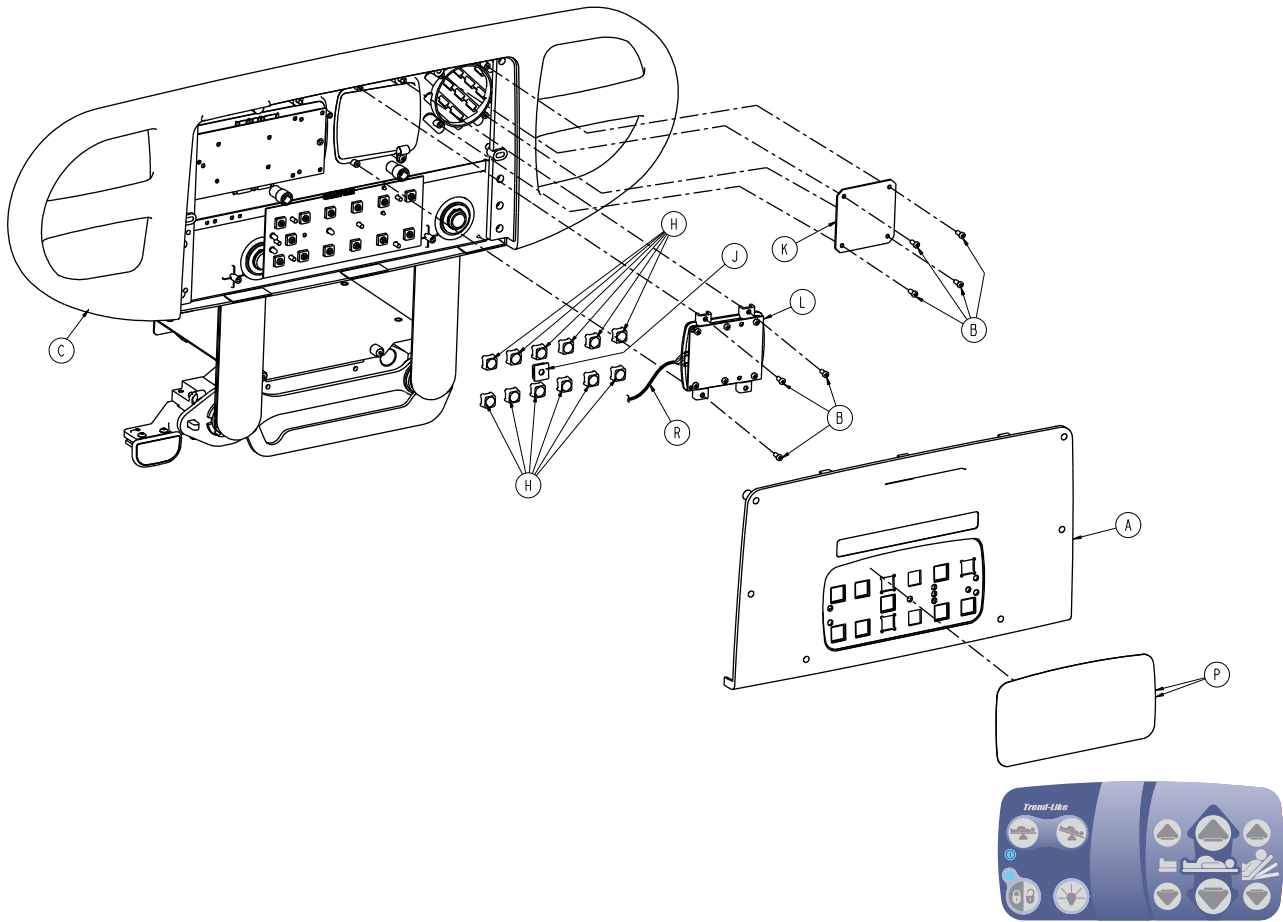

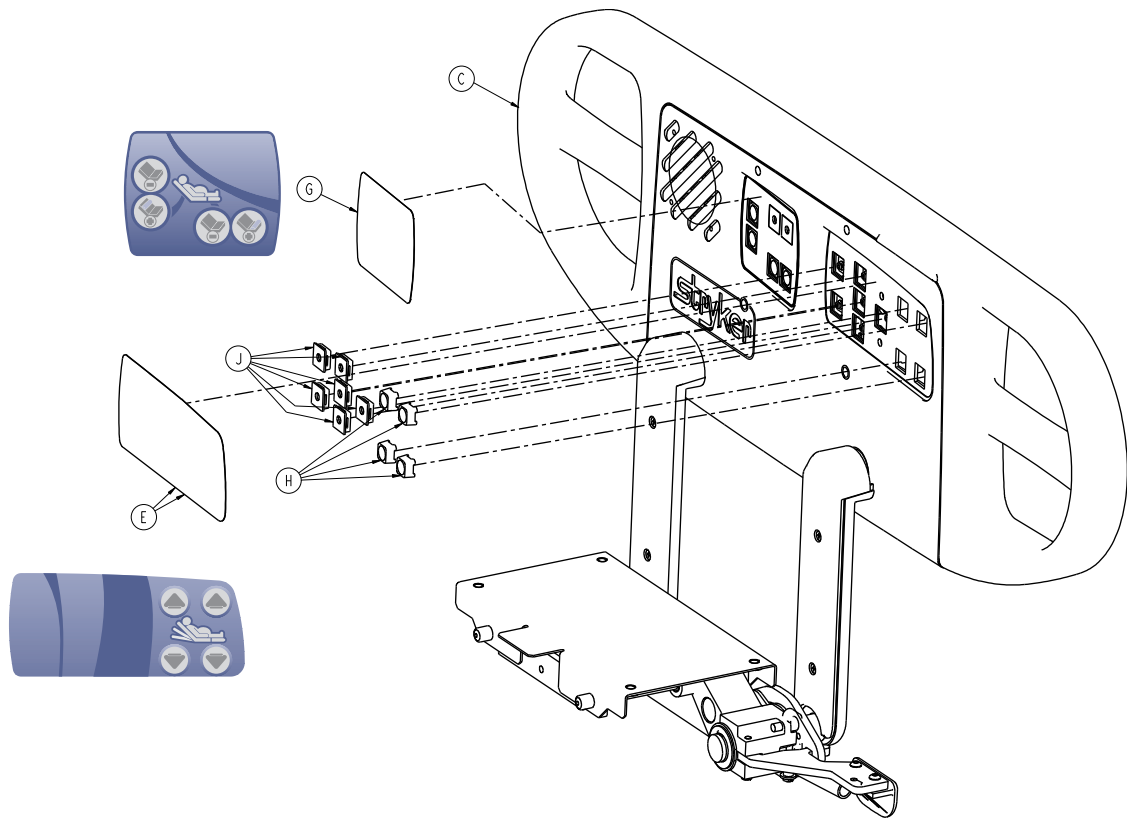



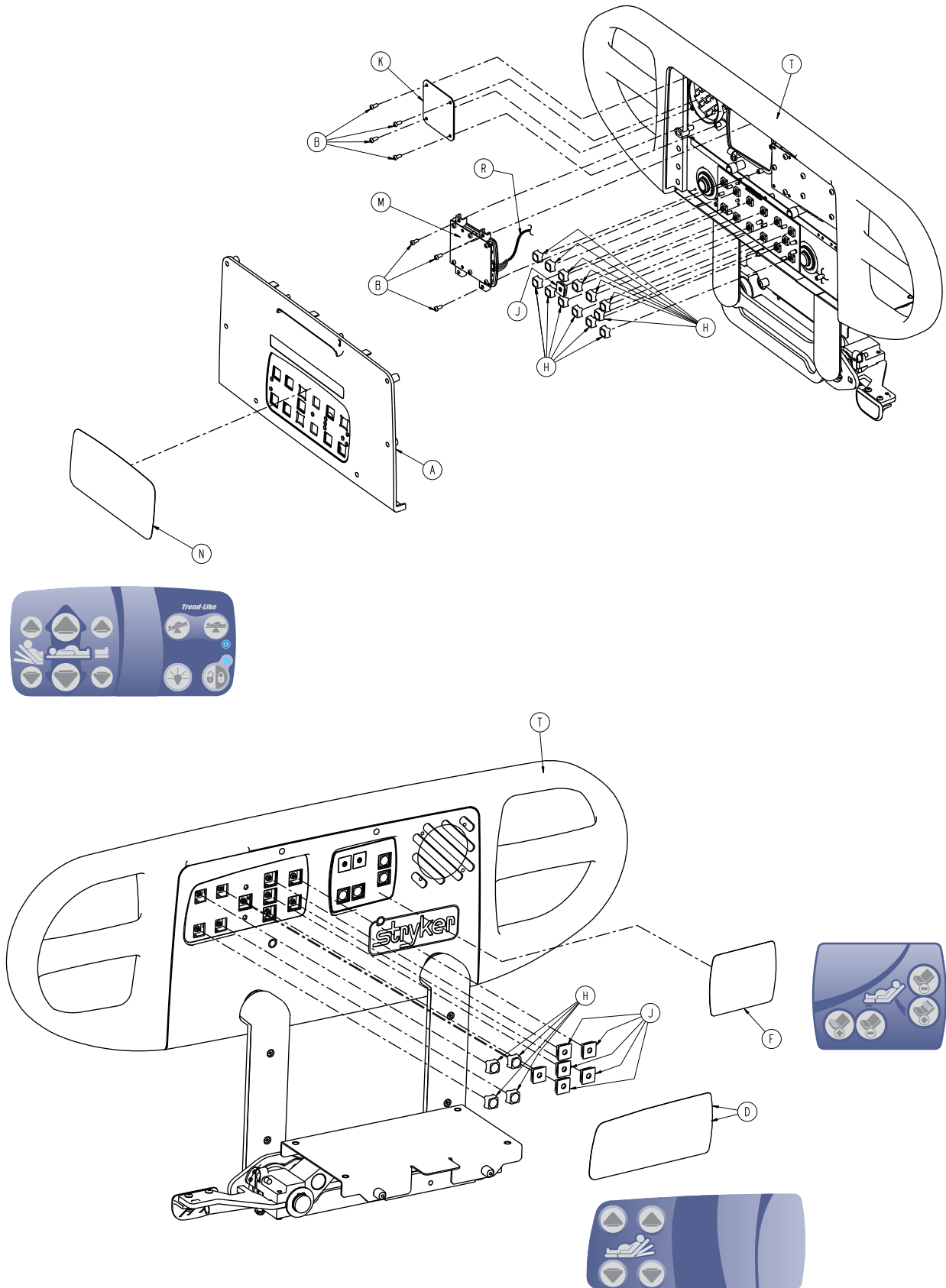



4701-120-070

A	4700-020-011	2
B	0023-088-000	14
C	4701-120-095	1
D	4701-109-928	1
E	4701-109-929	1
F	4701-109-933	1
G	4701-109-934	1
H	3001-400-953	12
J	3001-400-522	14
K	3001-400-517	2
L	5000-220-008	1
M	5000-220-010	1
N	4701-109-918	1
P	4701-109-919	1
R	5000-080-016	2
T	4701-120-100	1

Rev A

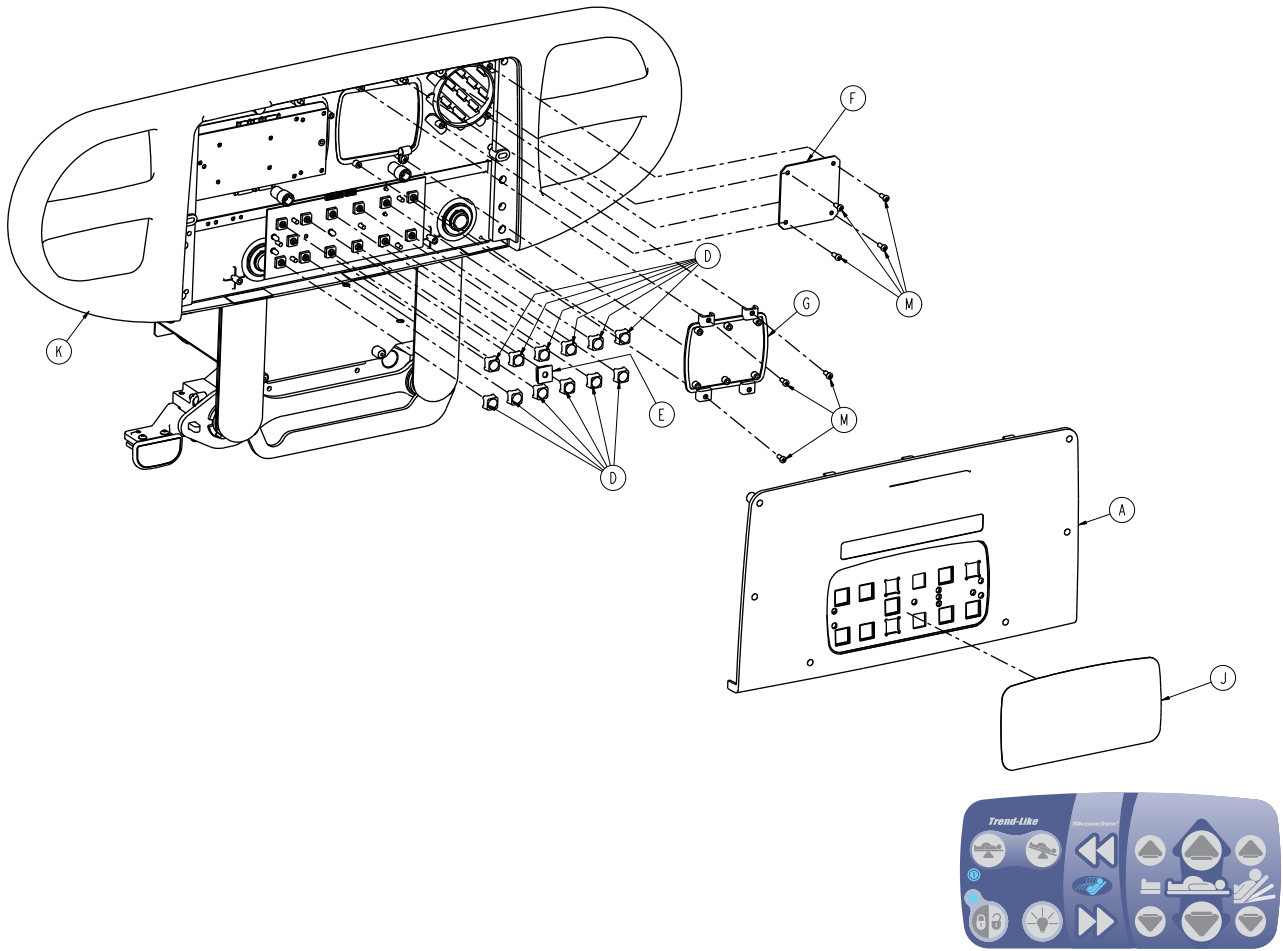


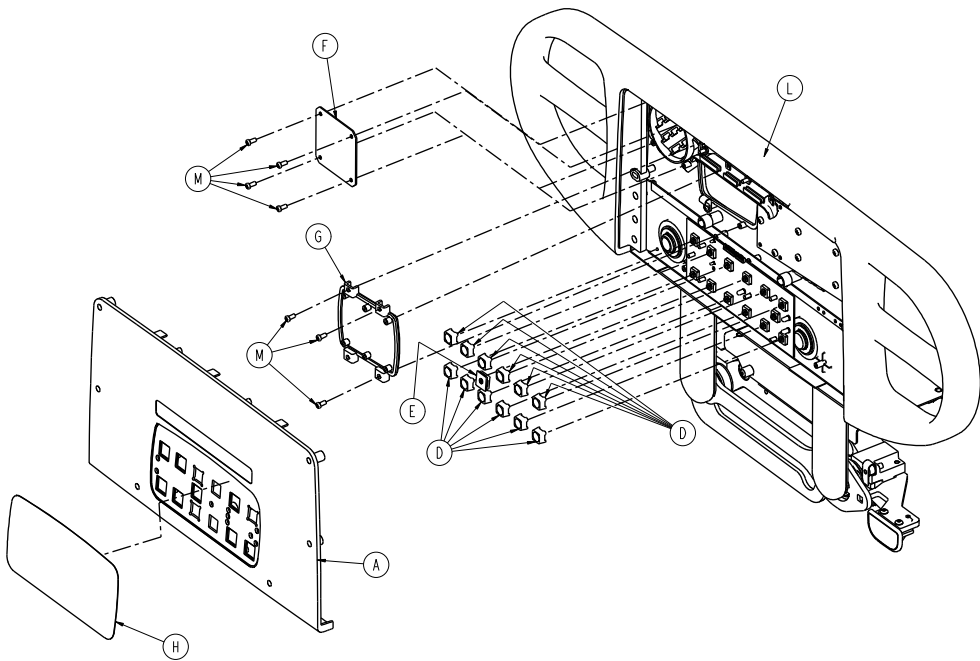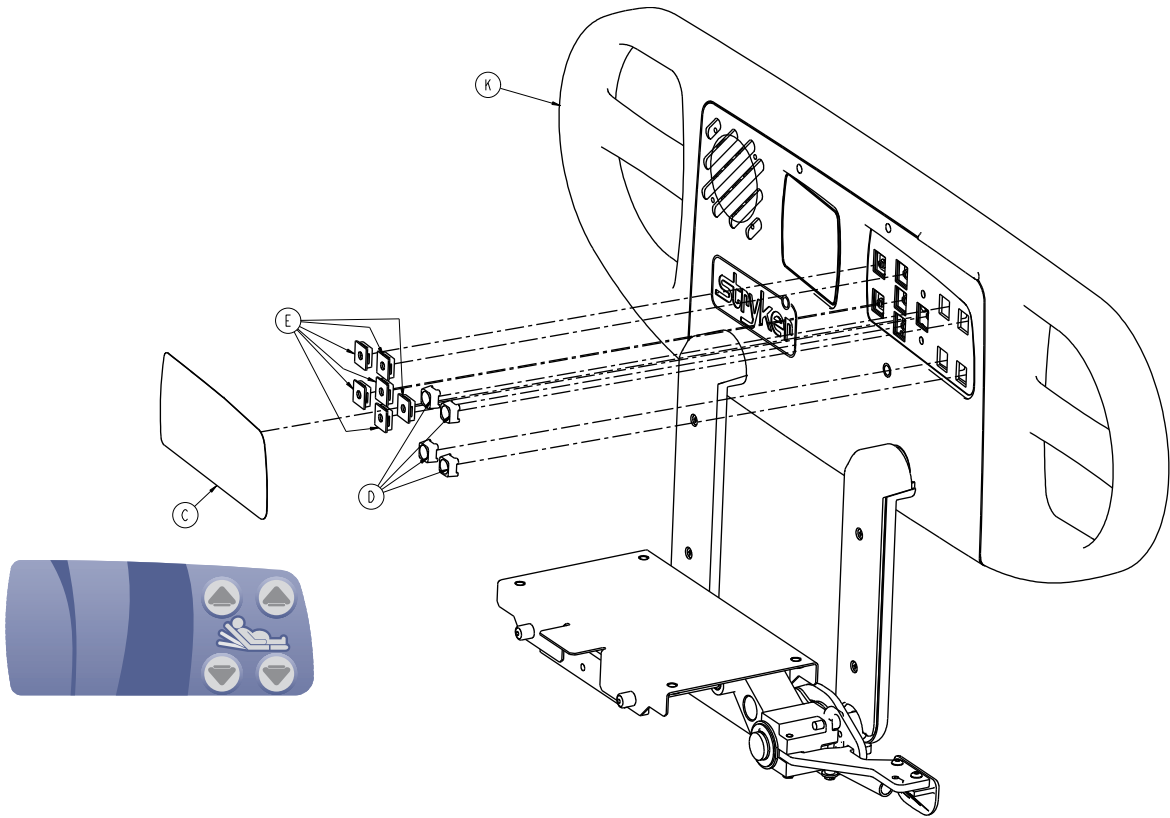


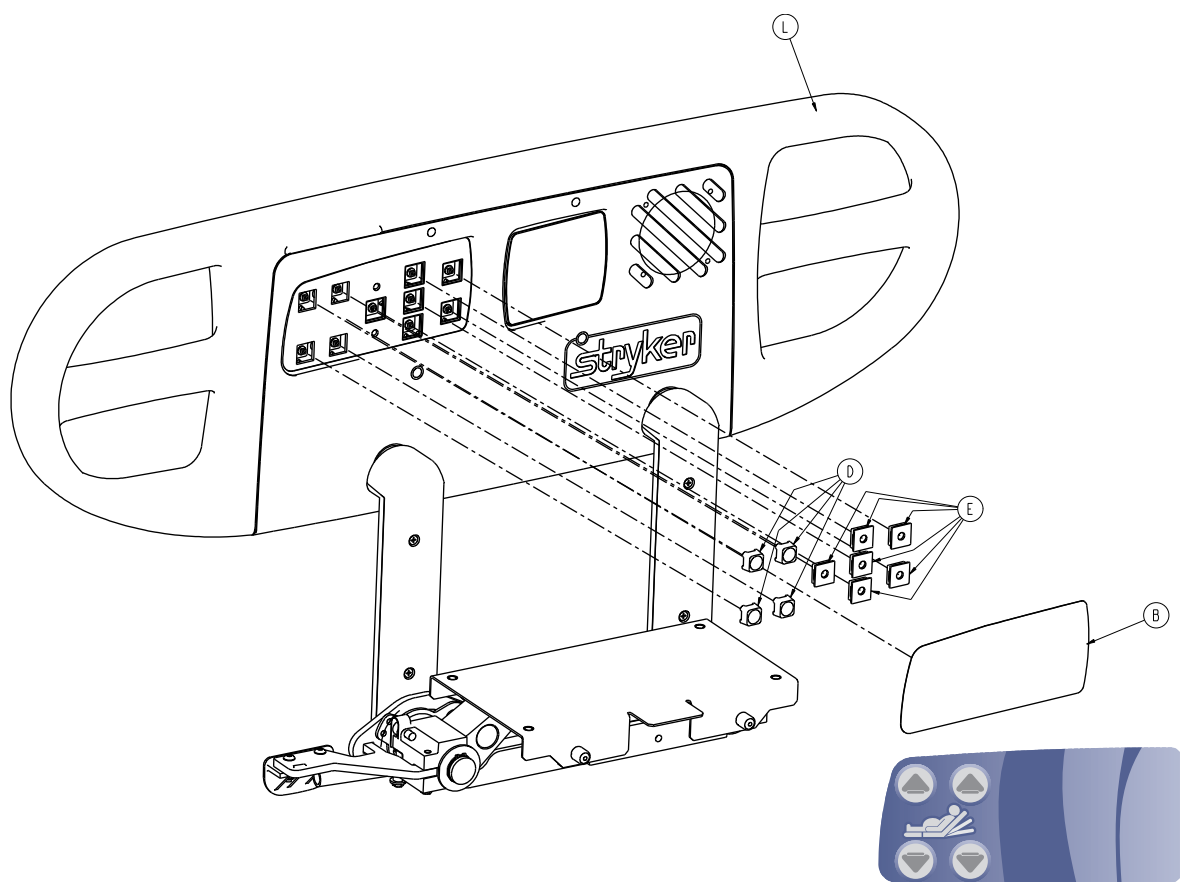


A	4700-020-011	2
B	4701-109-928	1
C	4701-109-929	1
D	3001-400-953	12
E	3001-400-522	14
F	3001-400-517	2
G	3001-400-535	2
H	4701-109-916	1
J	4701-109-917	1
K	4701-120-095	1
L	4701-120-100	1
M	0023-088-000	14

Rev A

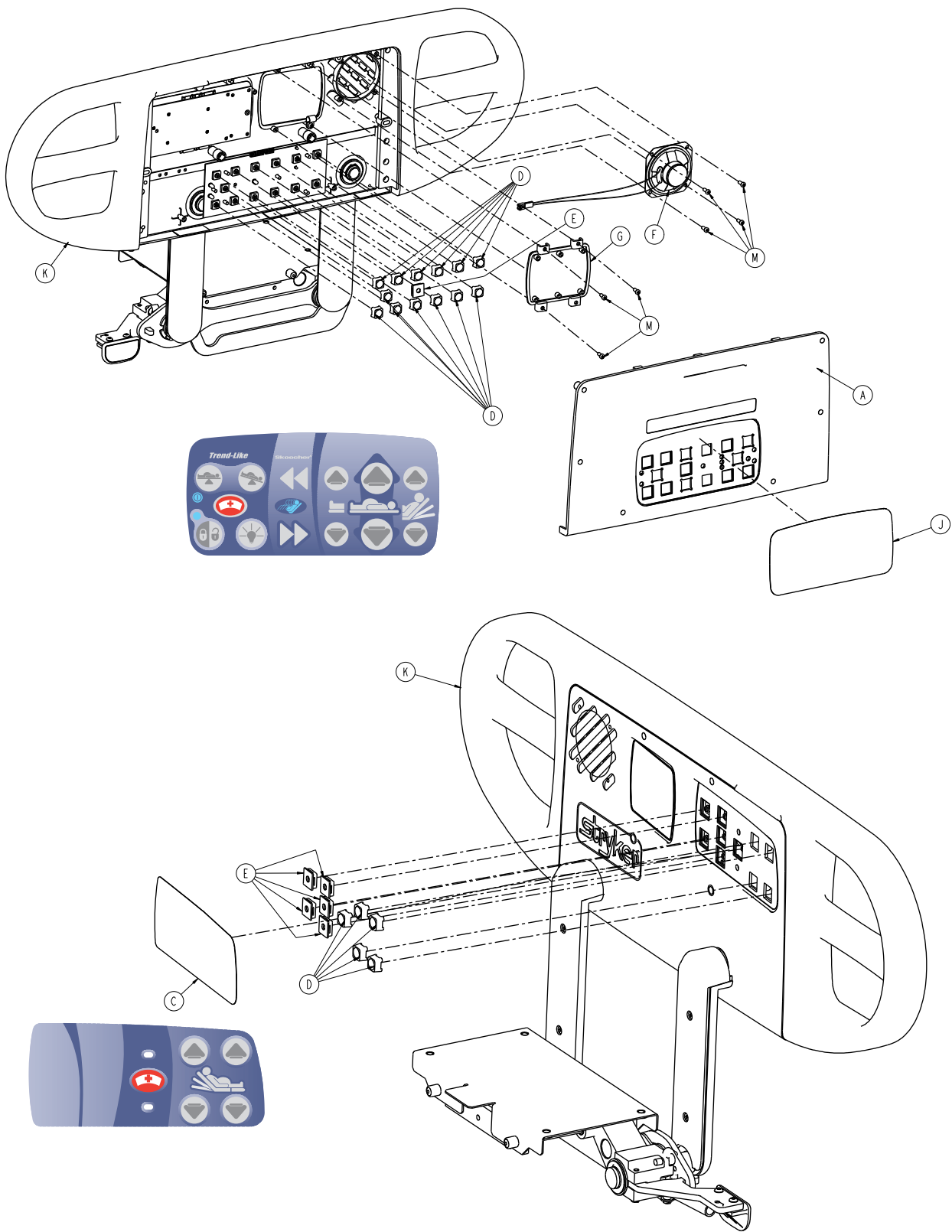


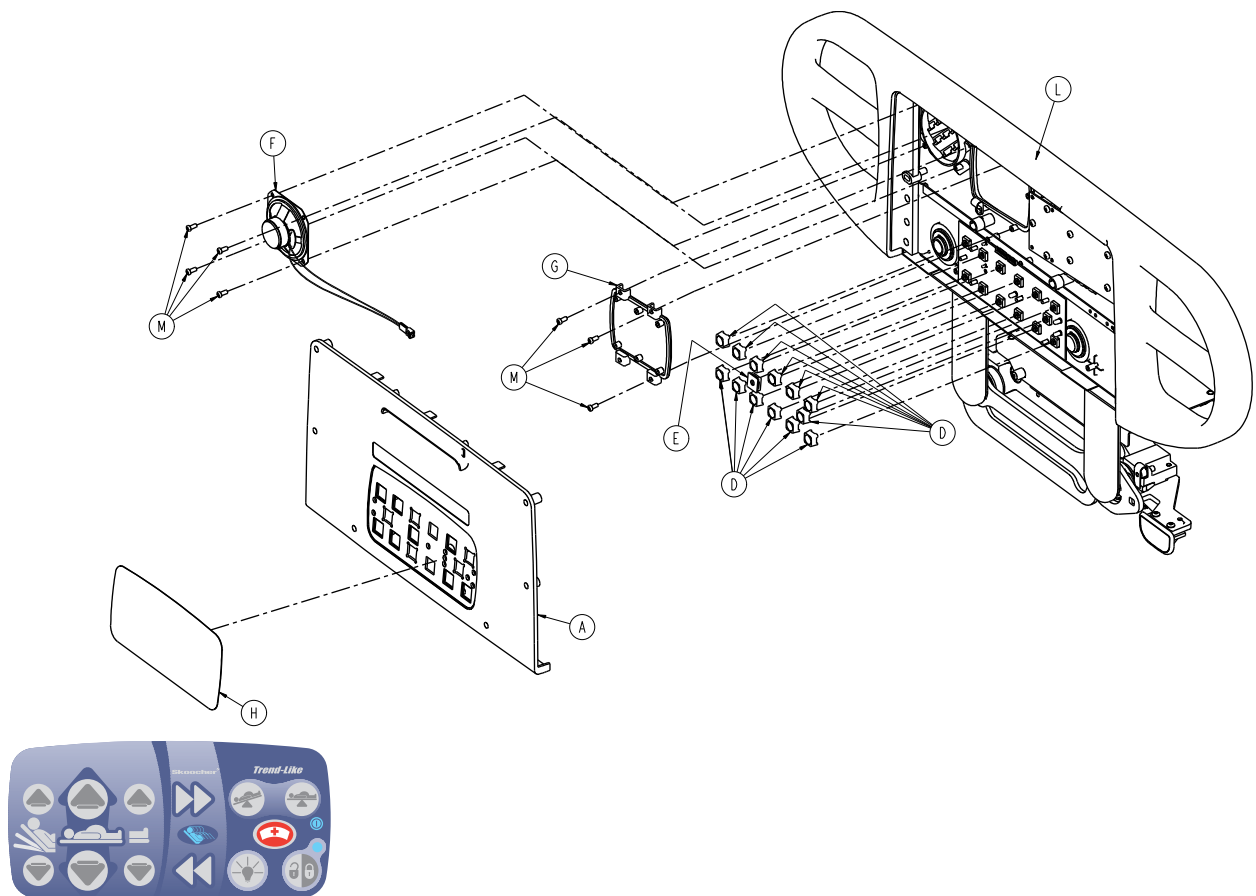


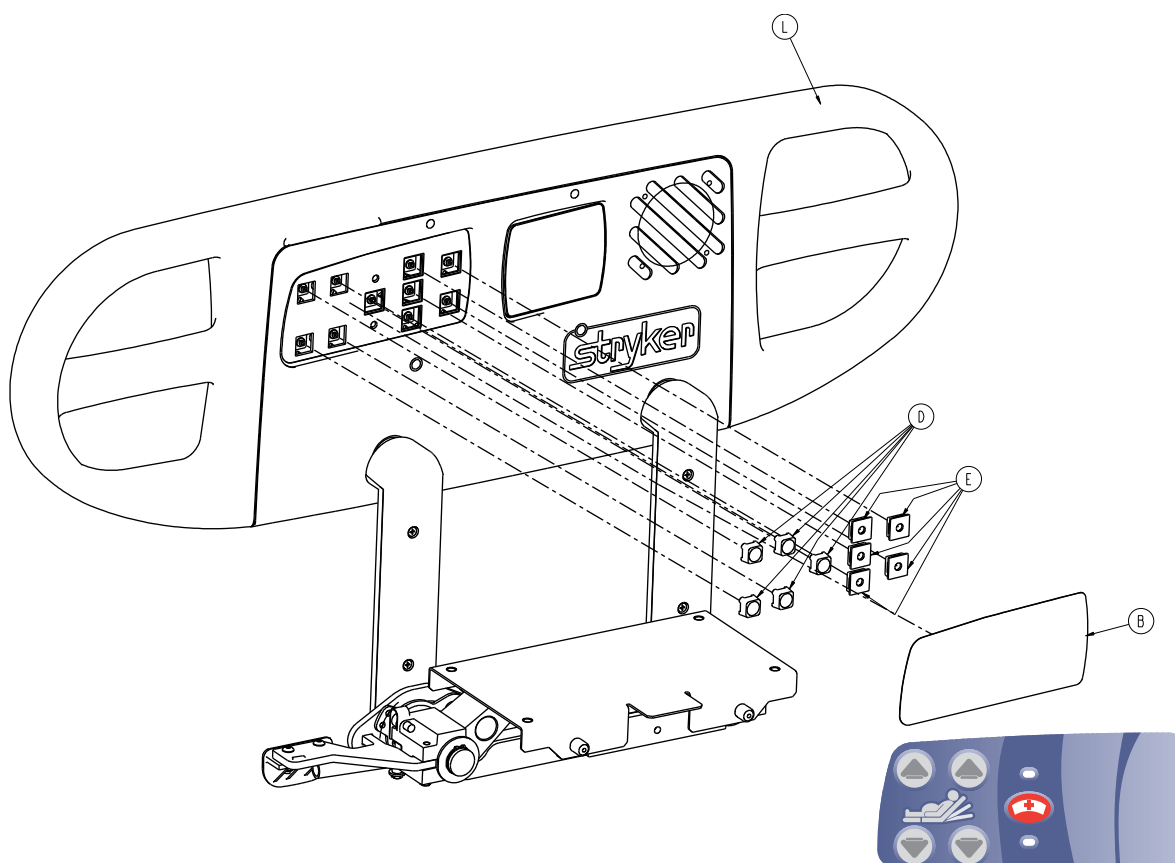


A	4700-020-009	2
B	4701-109-926	1
C	4701-109-927	1
D	3001-400-953	16
E	3001-400-522	12
F	3001-403-831	2
G	3001-400-535	2
H	4701-109-911	1
J	4701-109-912	1
K	4701-120-095	1
L	4701-120-100	1
M	0023-088-000	14

Rev A

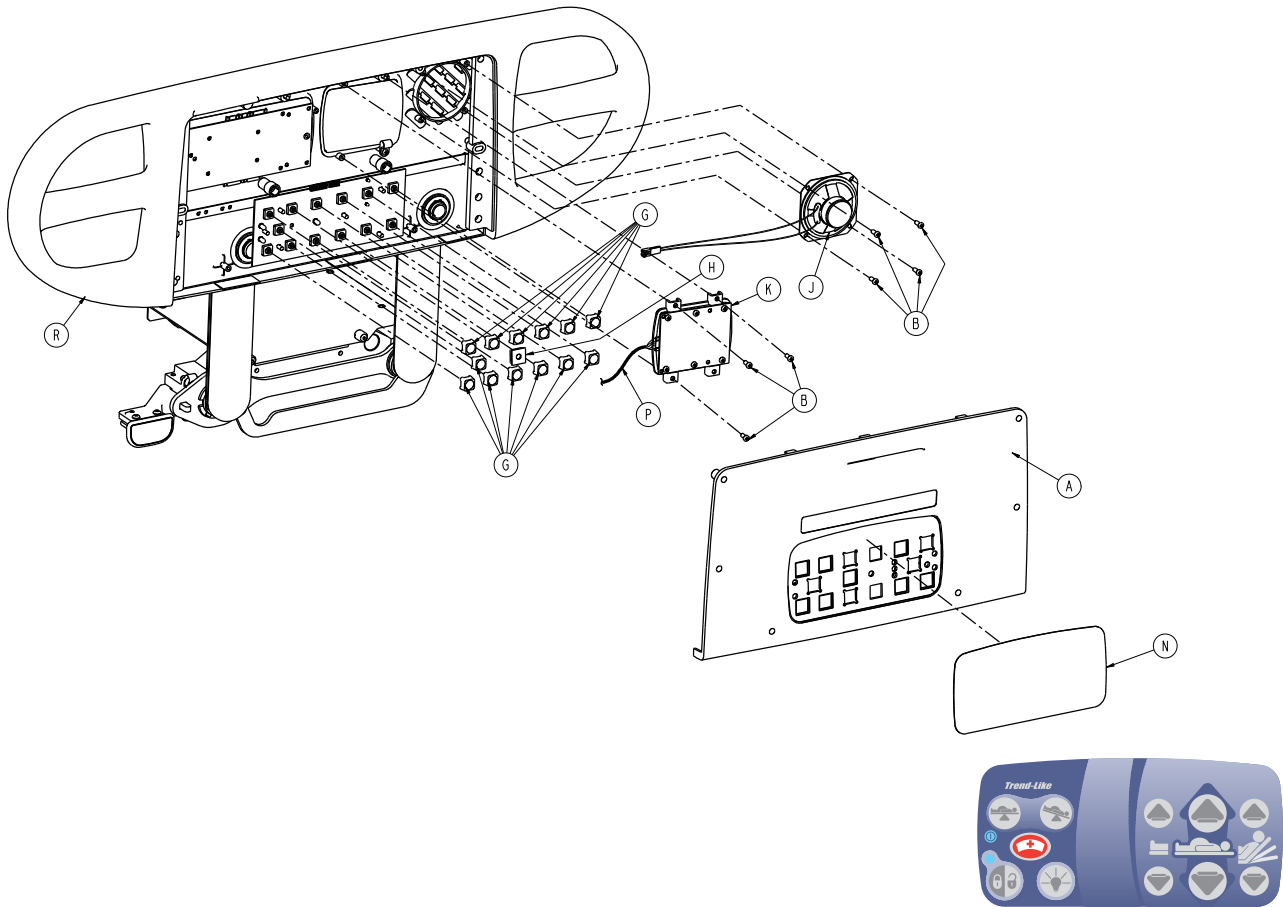

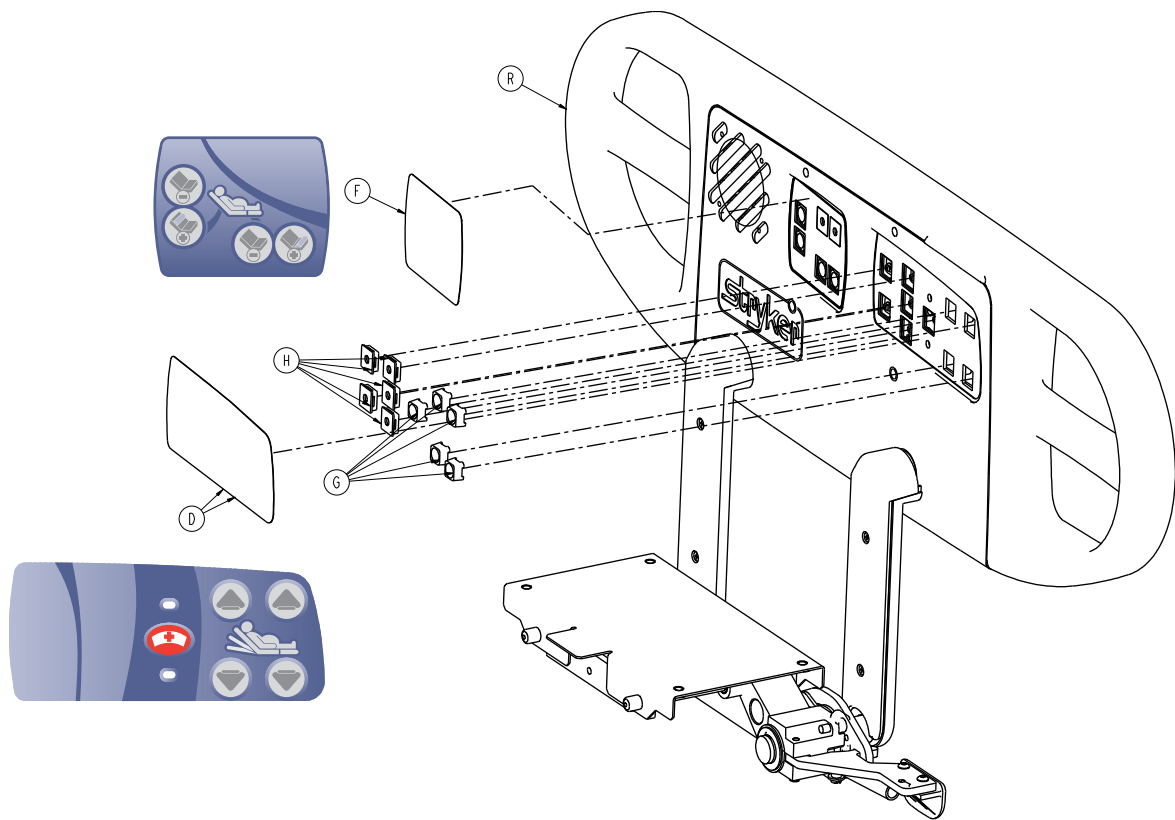


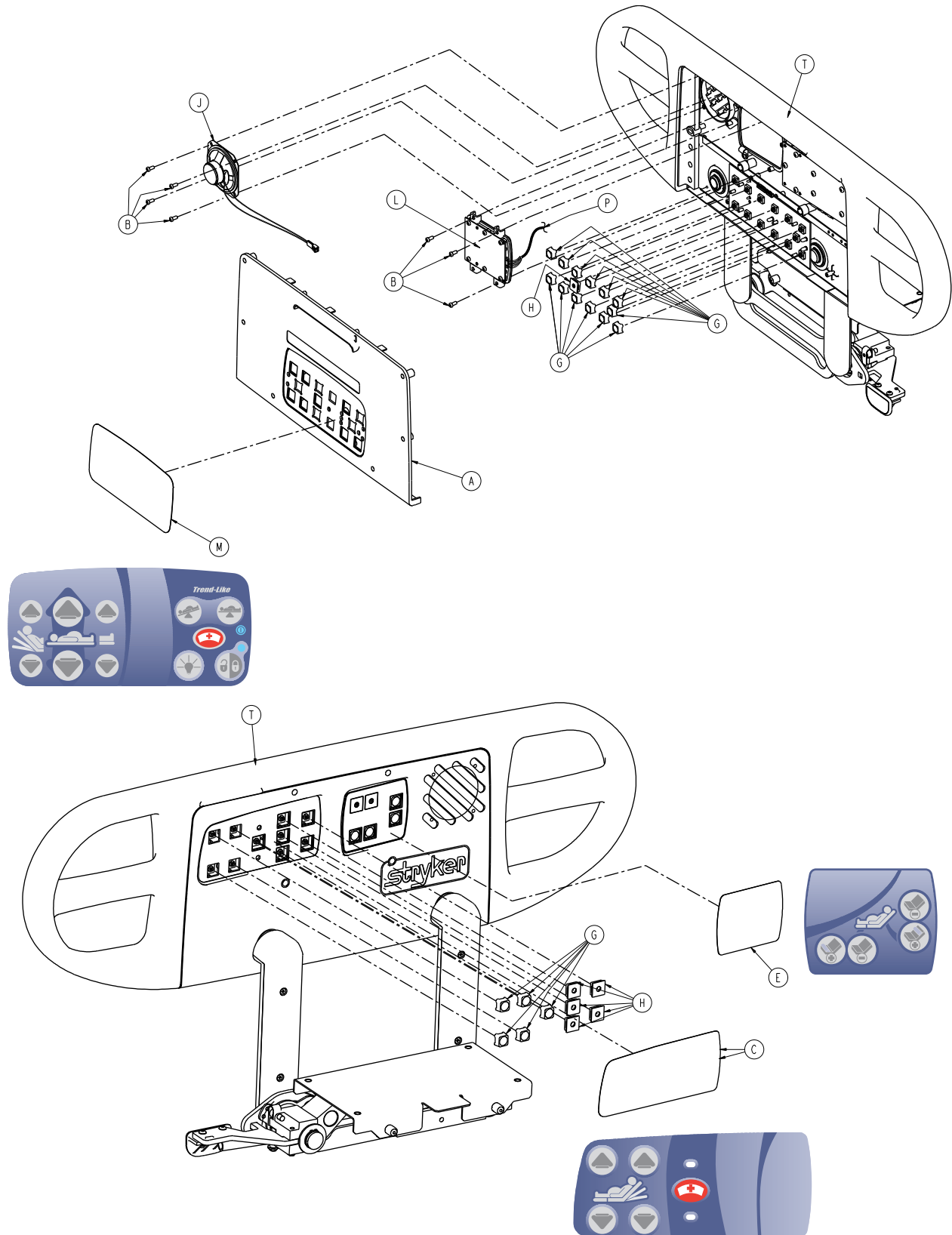



4701-120-050

A	4700-020-009	2
B	0023-088-000	14
C	4701-109-926	1
D	4701-109-927	1
E	4701-109-933	1
F	4701-109-934	1
G	3001-400-953	16
H	3001-400-522	12
J	3001-403-831	2
K	5000-220-008	1
L	5000-220-010	1
M	4701-109-913	1
N	4701-109-914	1
P	5000-080-016	2
R	4701-120-095	1
T	4701-120-100	1

Rev A

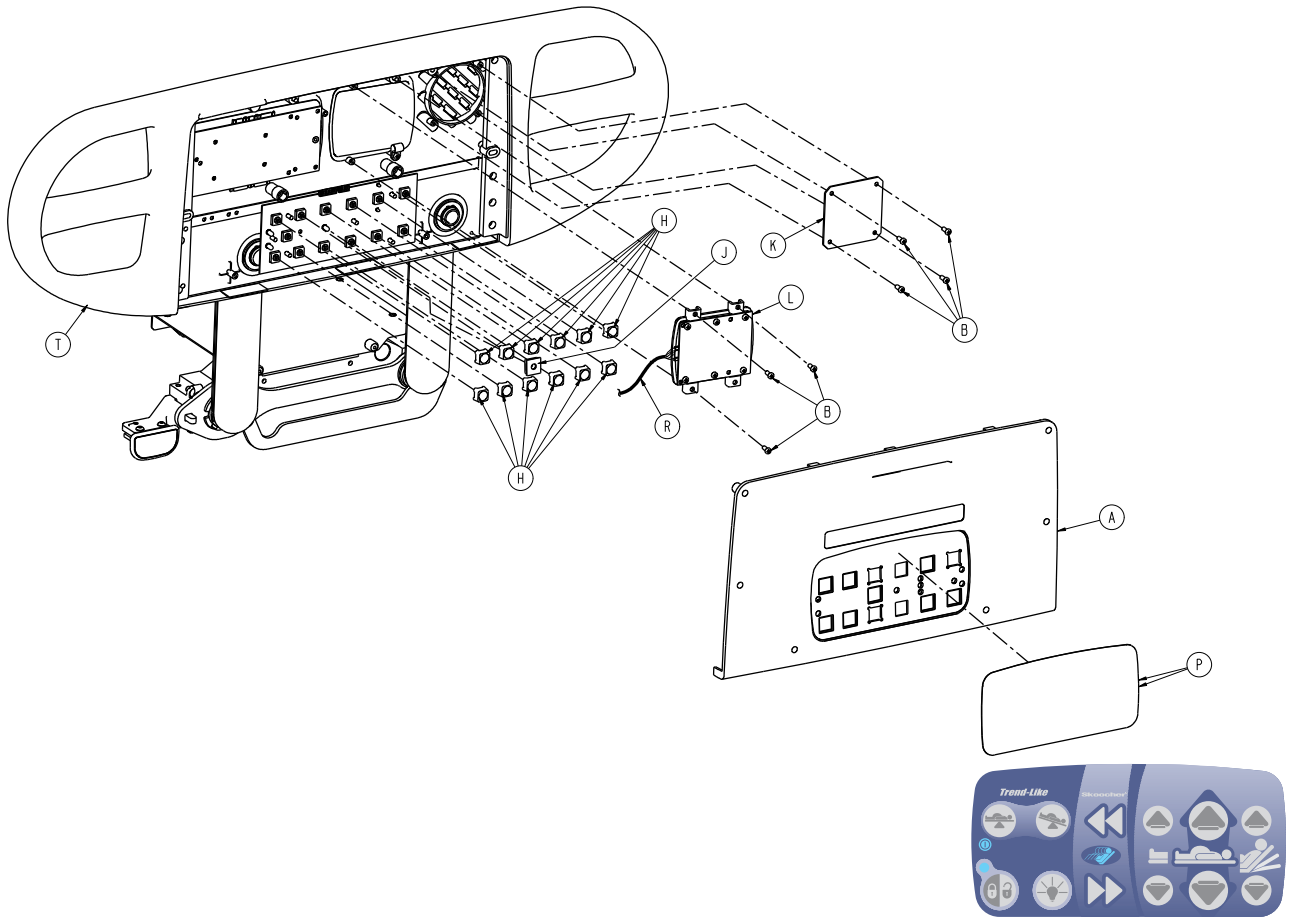


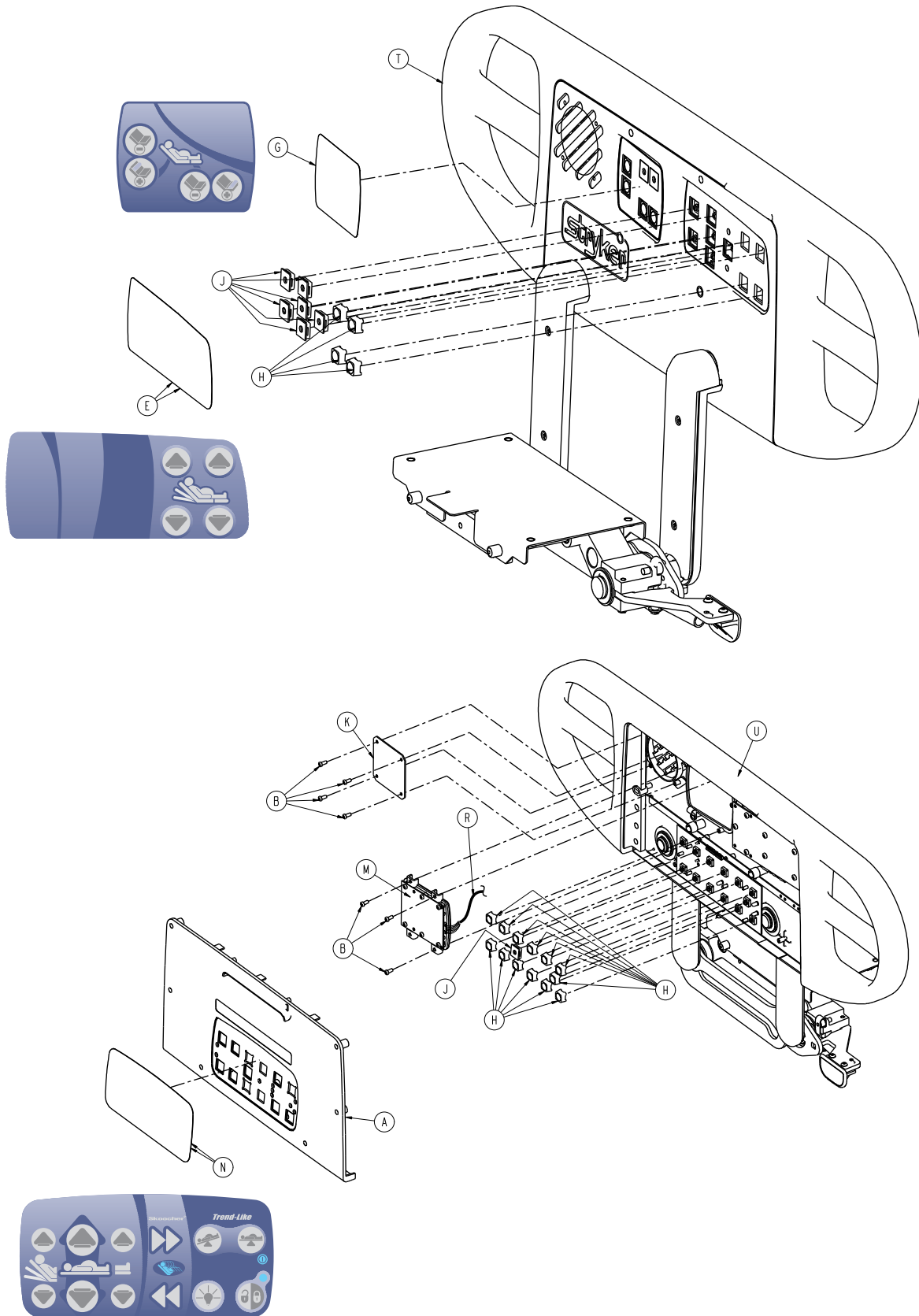


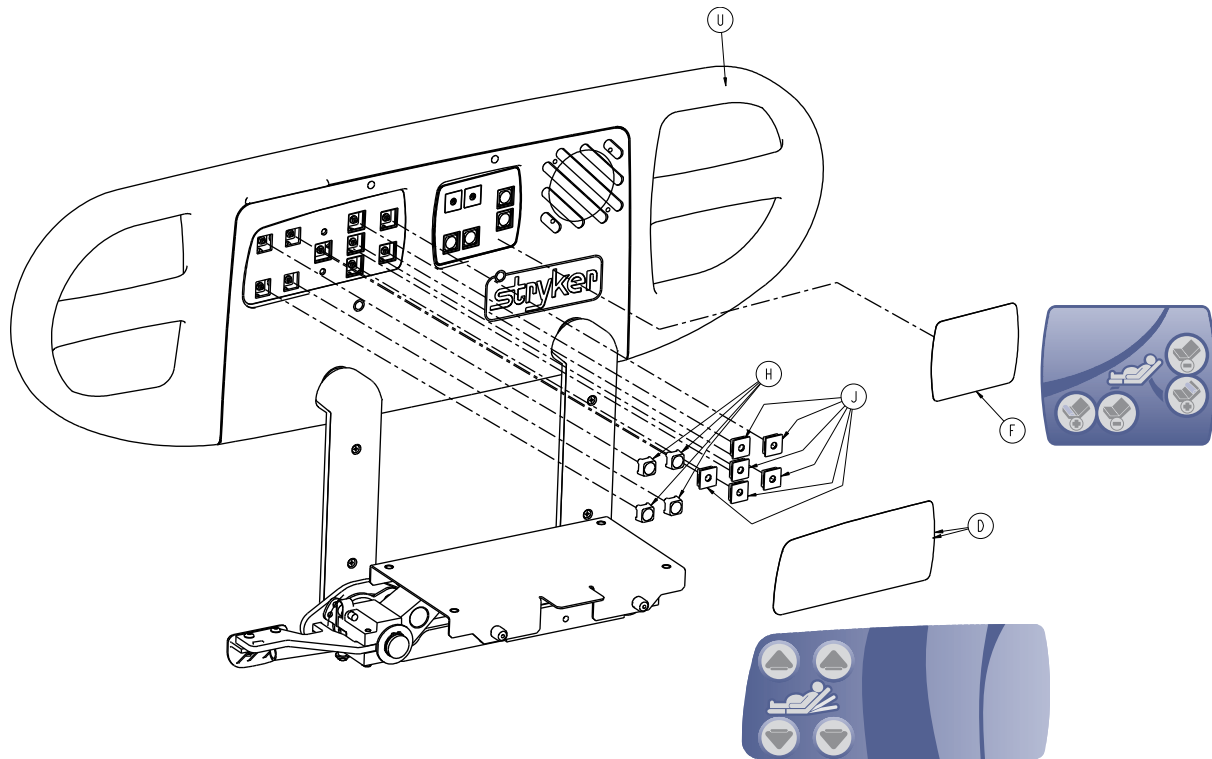


A	4700-020-011	2
B	0023-088-000	14
C	4701-120-100	1
D	4701-109-928	1
E	4701-109-929	1
F	4701-109-933	1
G	4701-109-934	1
H	3001-400-953	12
J	3001-400-522	14
K	3001-400-517	2
L	5000-220-008	1
M	5000-220-010	1
N	4701-109-916	1
P	4701-109-917	1
R	5000-080-016	2
T	4701-120-095	1


Rev A

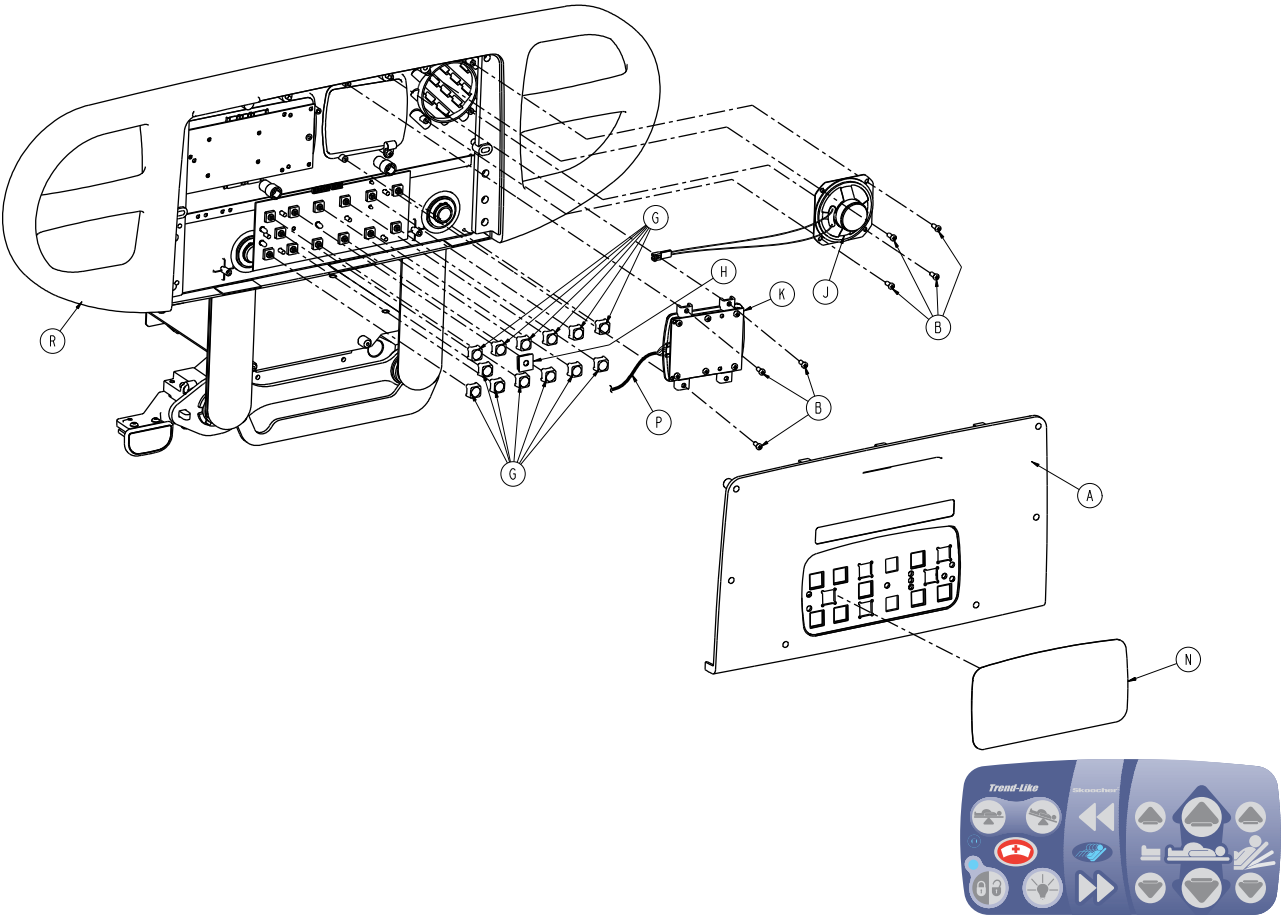


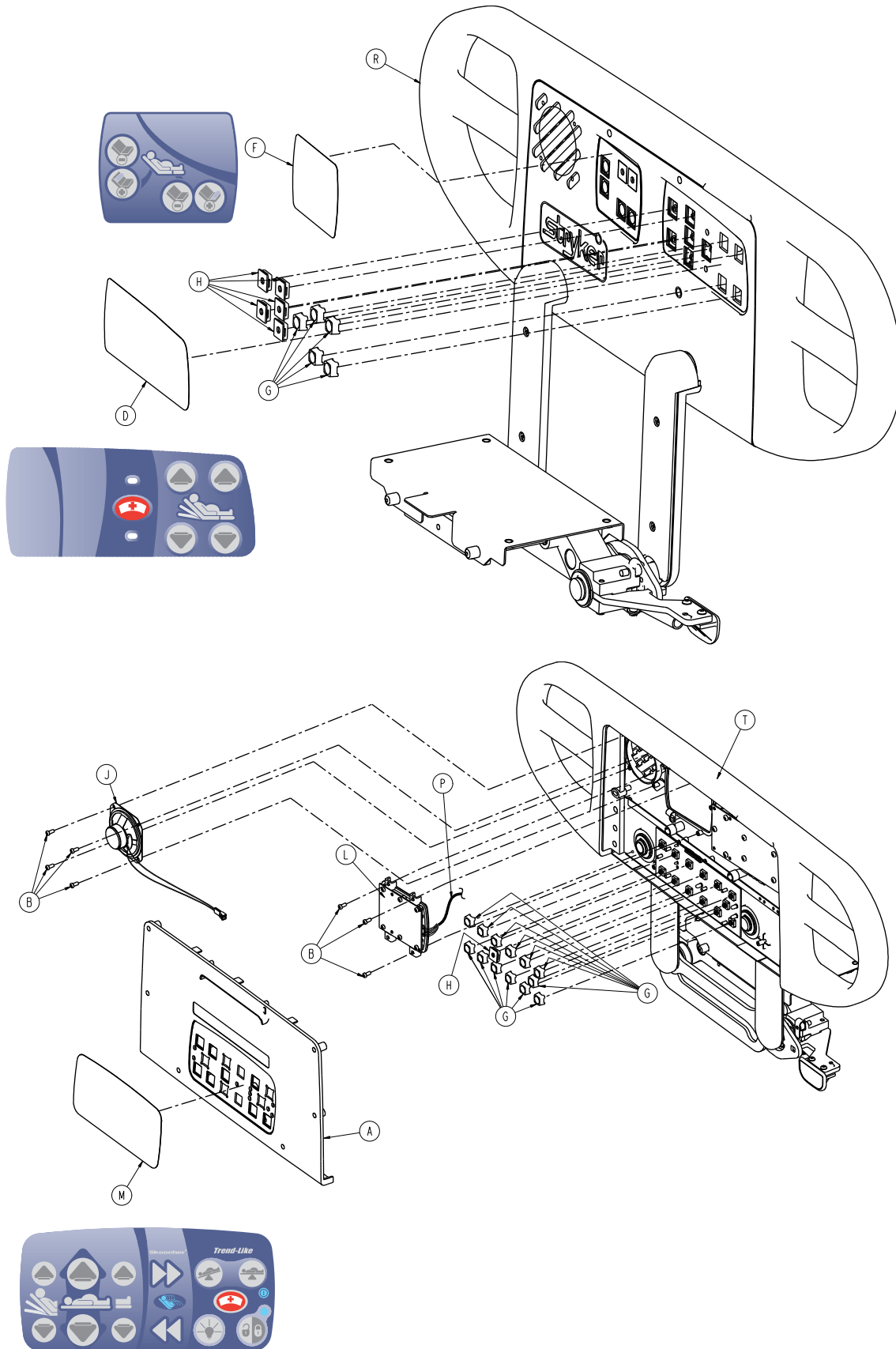


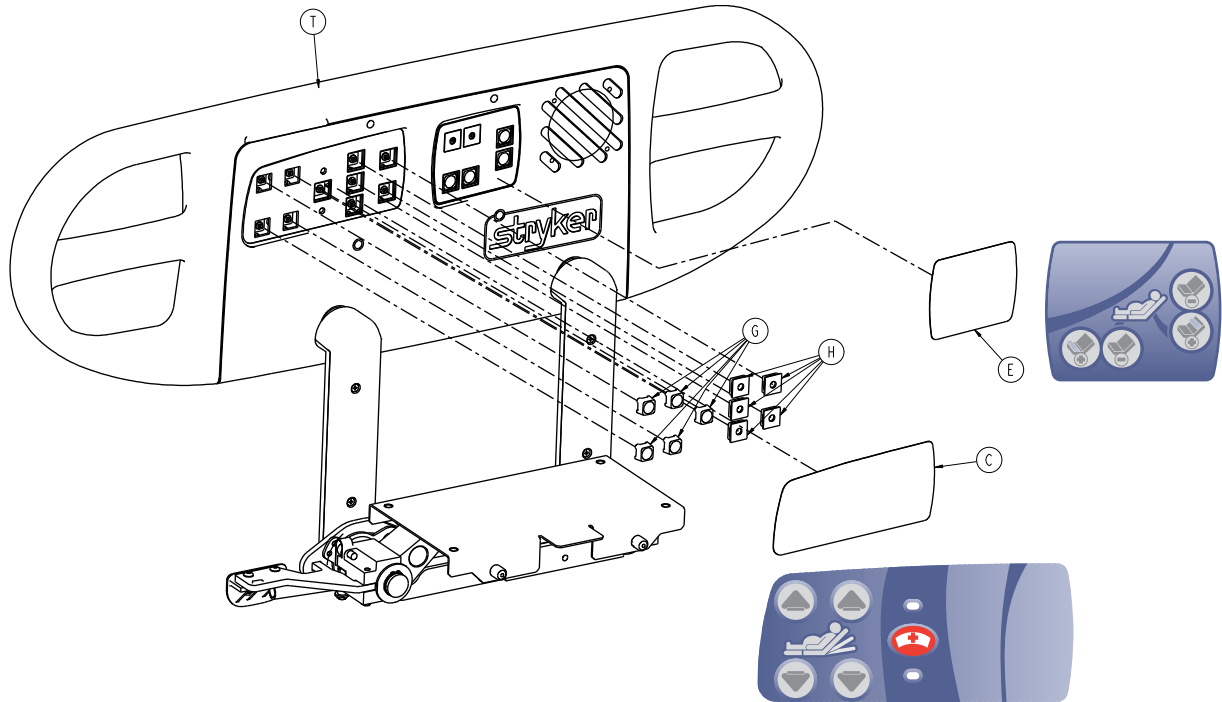


A	4700-020-009	2
B	0023-088-000	14
C	4701-109-926	1
D	4701-109-927	1
E	4701-109-933	1
F	4701-109-934	1
G	3001-400-953	16
H	3001-400-522	12
J	3001-403-831	2
K	5000-220-008	1
L	5000-220-010	1
M	4701-109-911	1
N	4701-109-912	1
P	5000-080-016	2
R	4701-120-095	1
T	4701-120-100	1

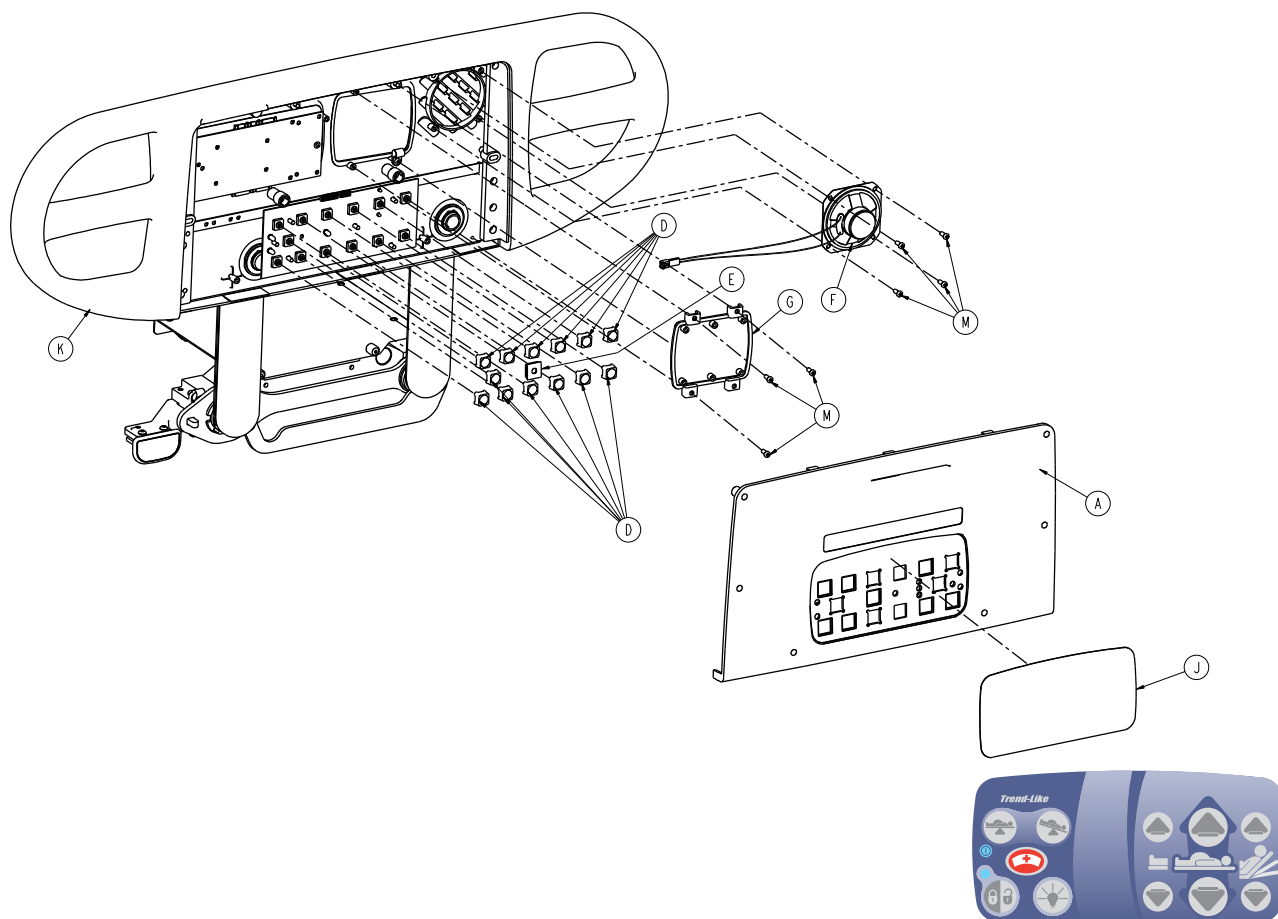

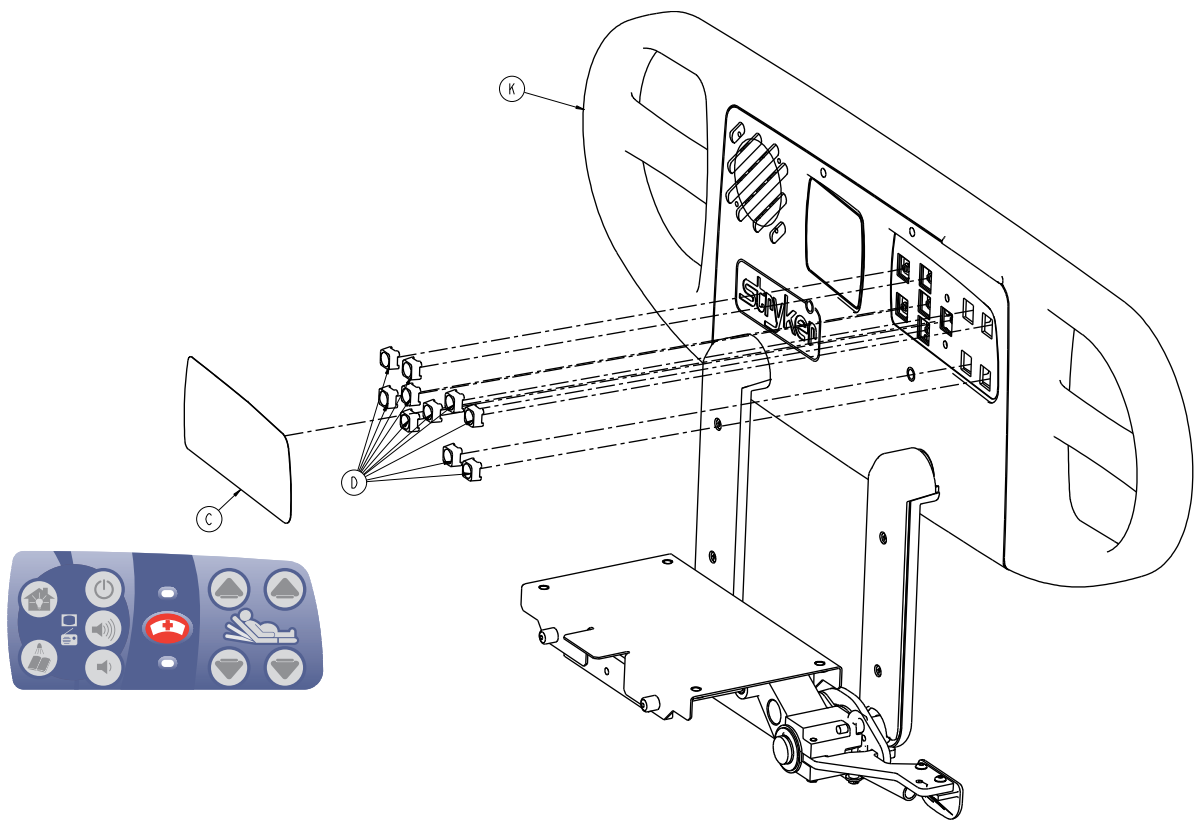


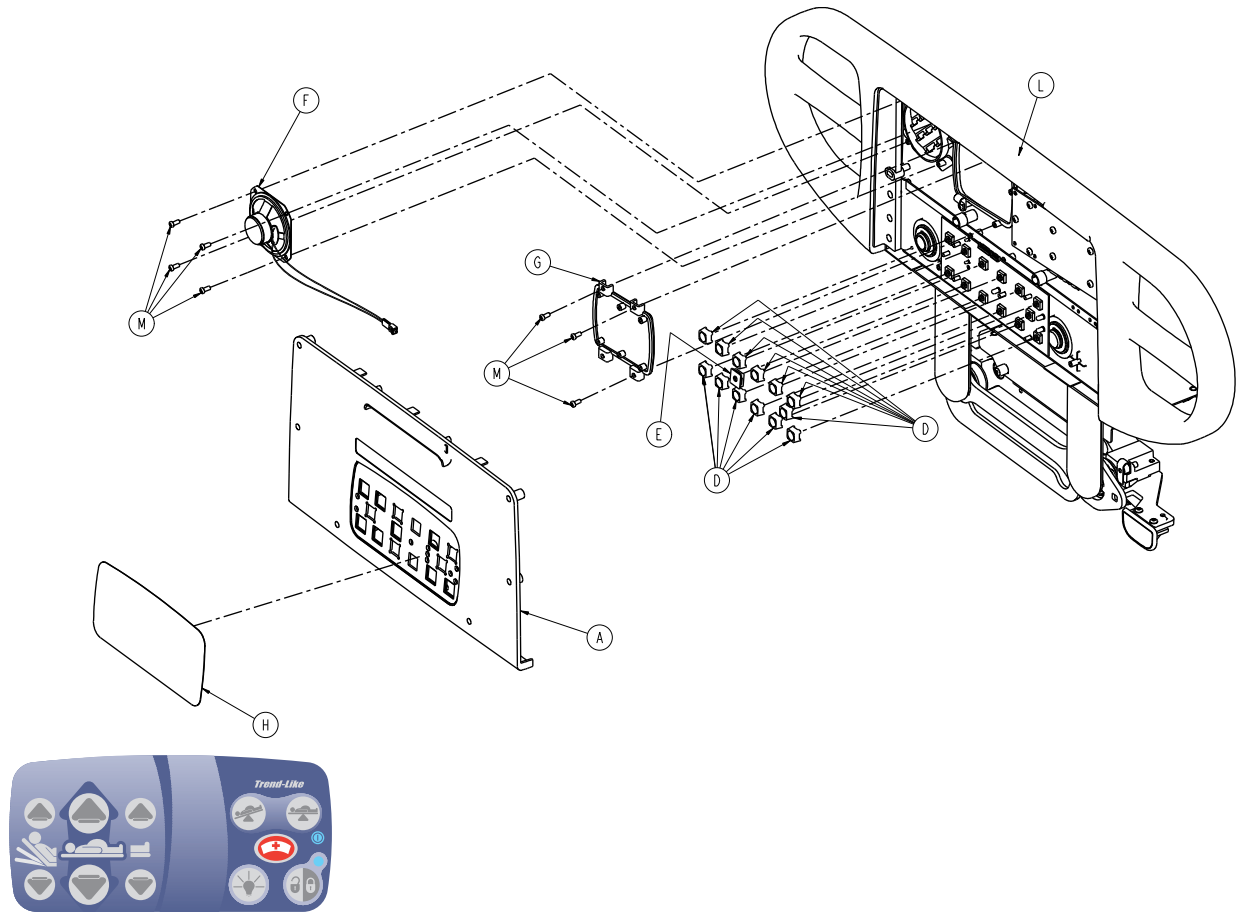


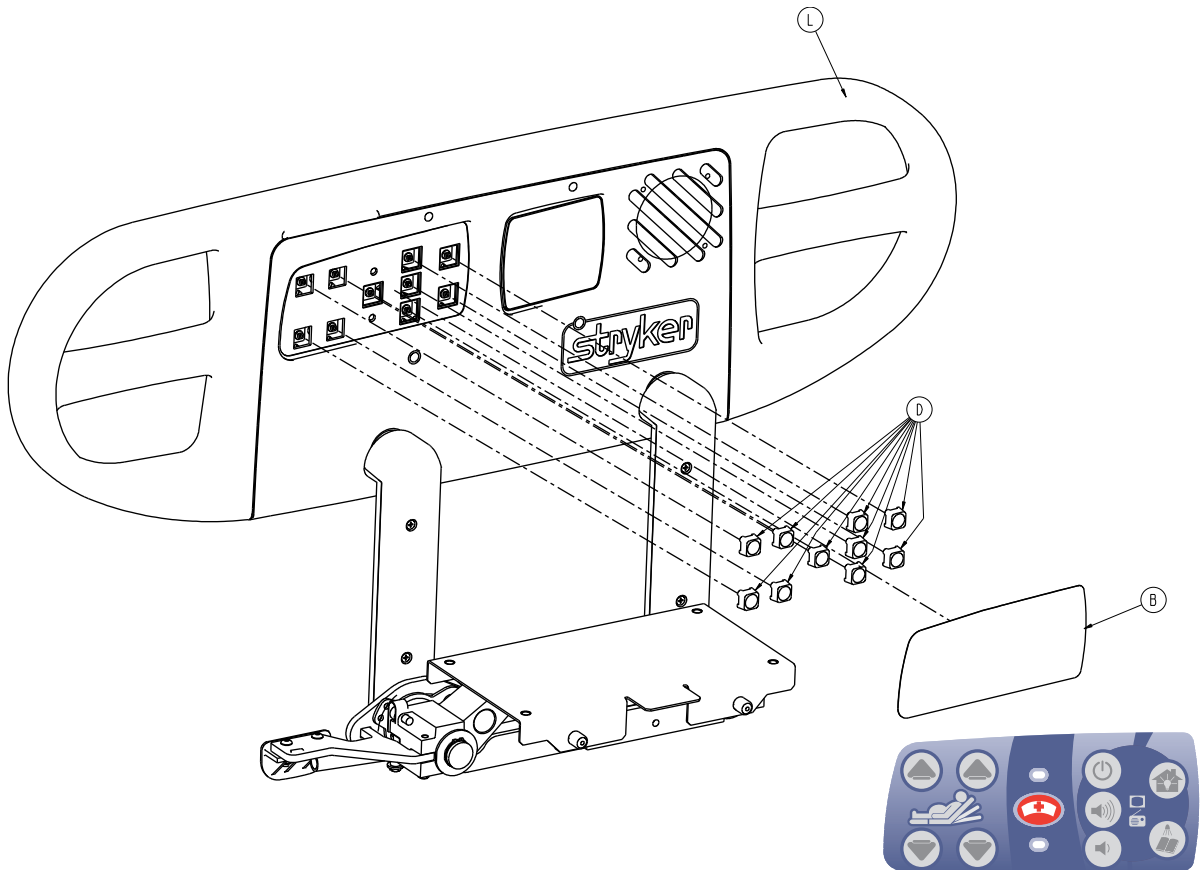

A	4700-020-009	2
B	4701-109-923	1
C	4701-109-924	1
D	3001-400-953	26
E	3001-400-522	2
F	3001-403-831	2
G	3001-400-535	2
H	4701-109-913	1
J	4701-109-914	1
K	4701-120-095	1
L	4701-120-100	1
M	0023-088-000	14

Rev A

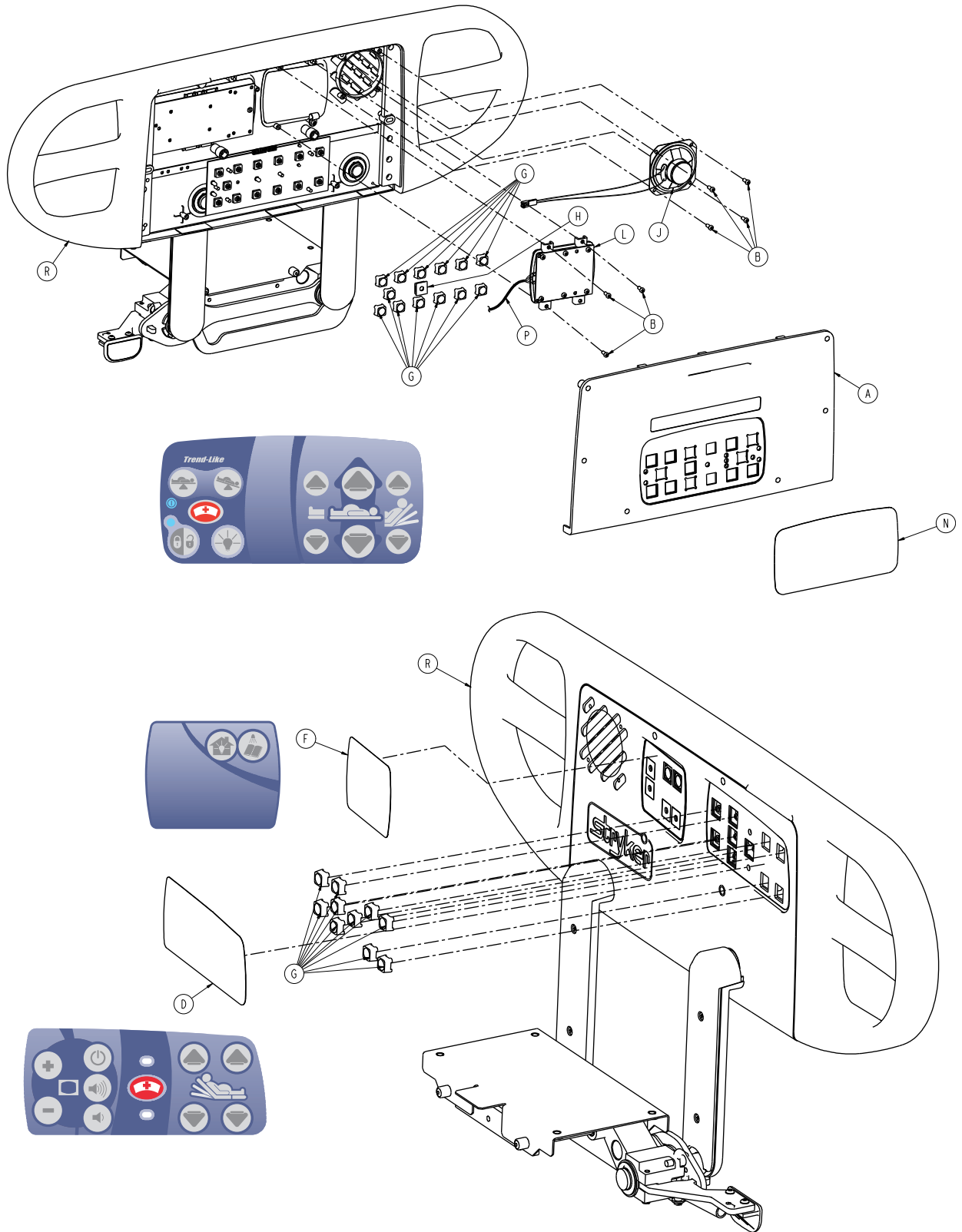


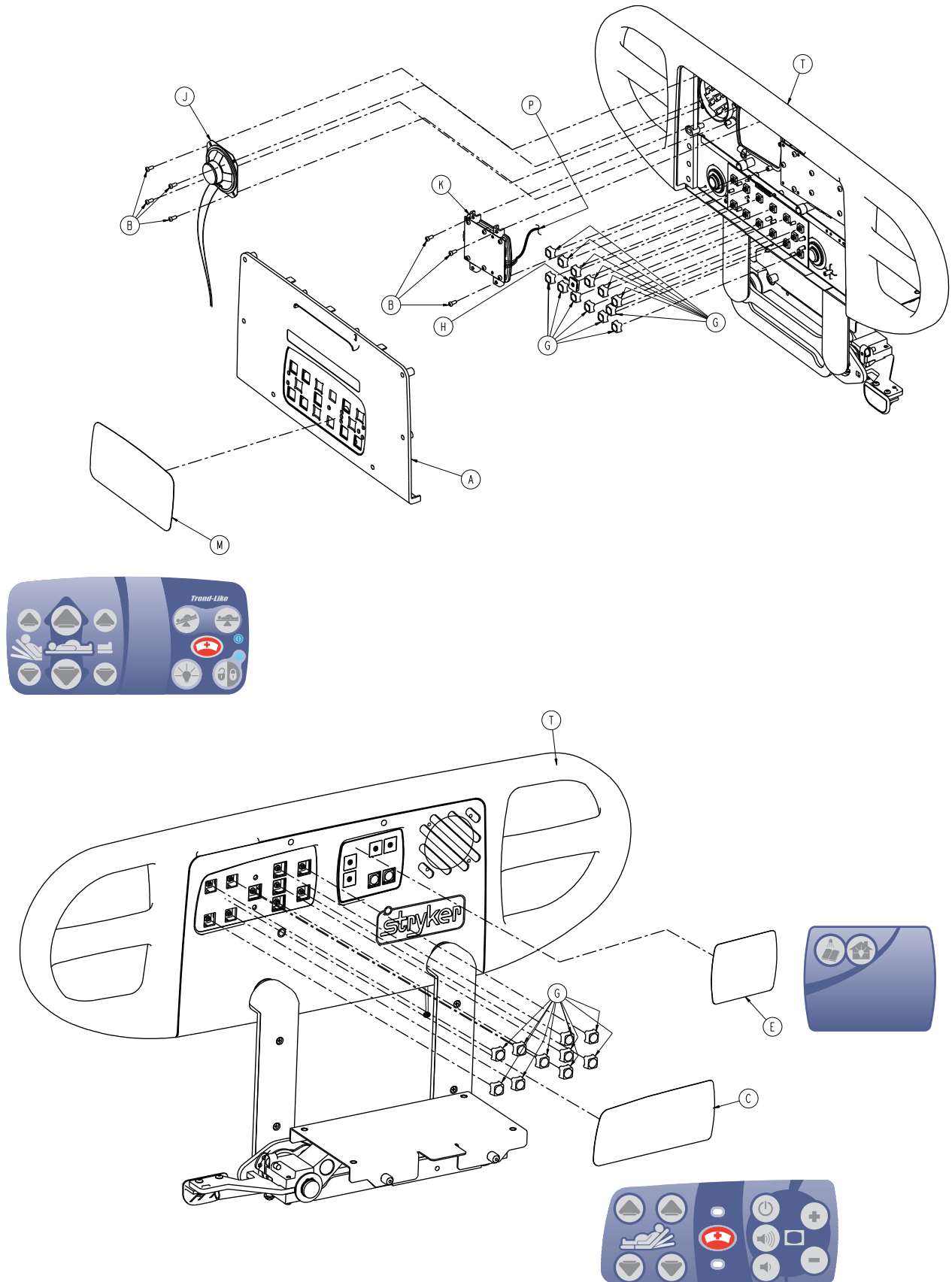


4701-120-025

A	4700-020-009	2
B	0023-088-000	14
C	4701-109-921	1
D	4701-109-922	1
E	4701-109-936	1
F	4701-109-937	1
G	3001-400-953	26
H	3001-400-522	2
J	3001-403-831	2
K	5000-020-028	1
L	5000-020-029	1
M	4701-109-913	1
N	4701-109-914	1
P	5000-080-016	2
R	4701-120-095	1
T	4701-120-100	1


Rev A

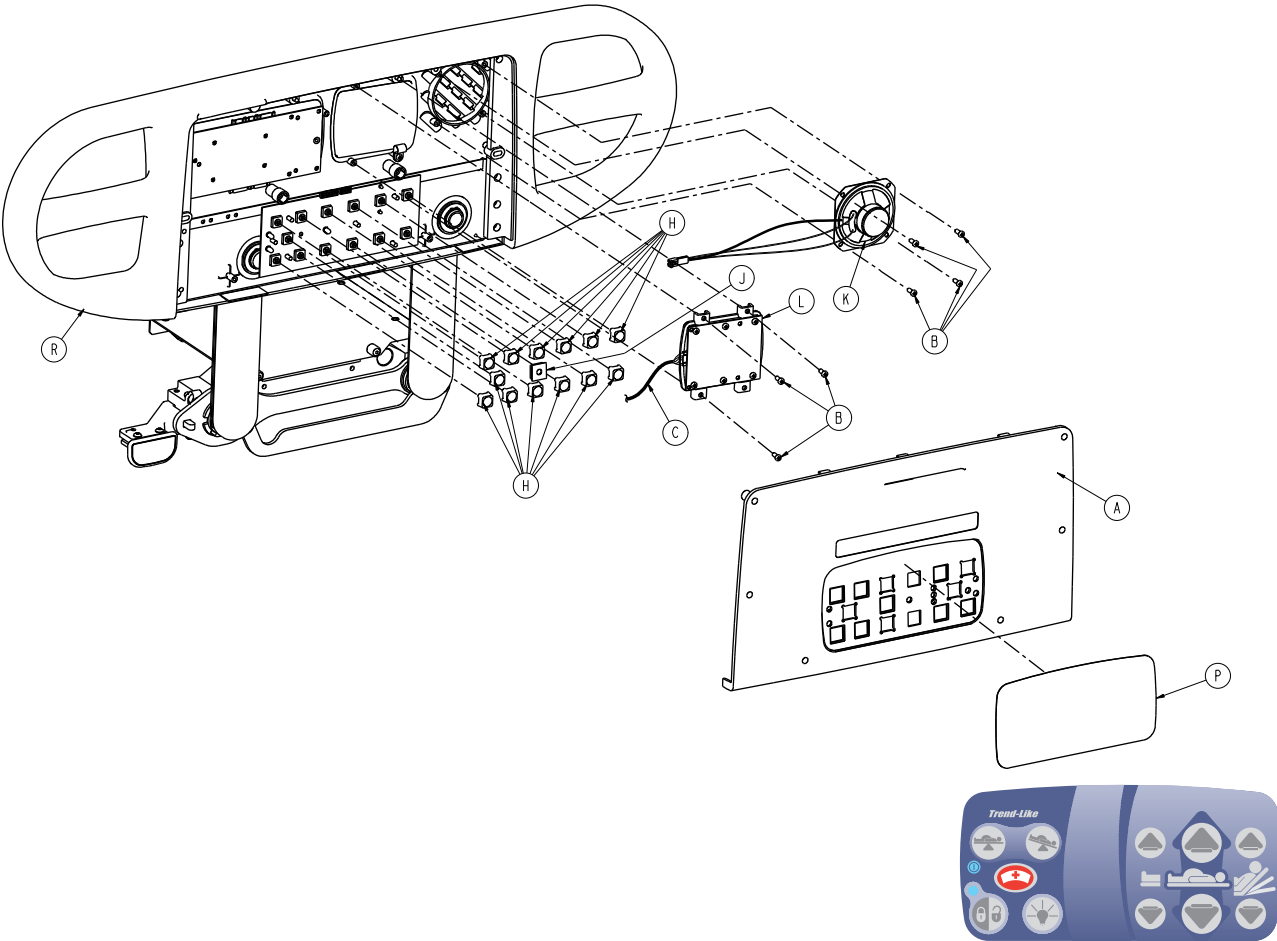


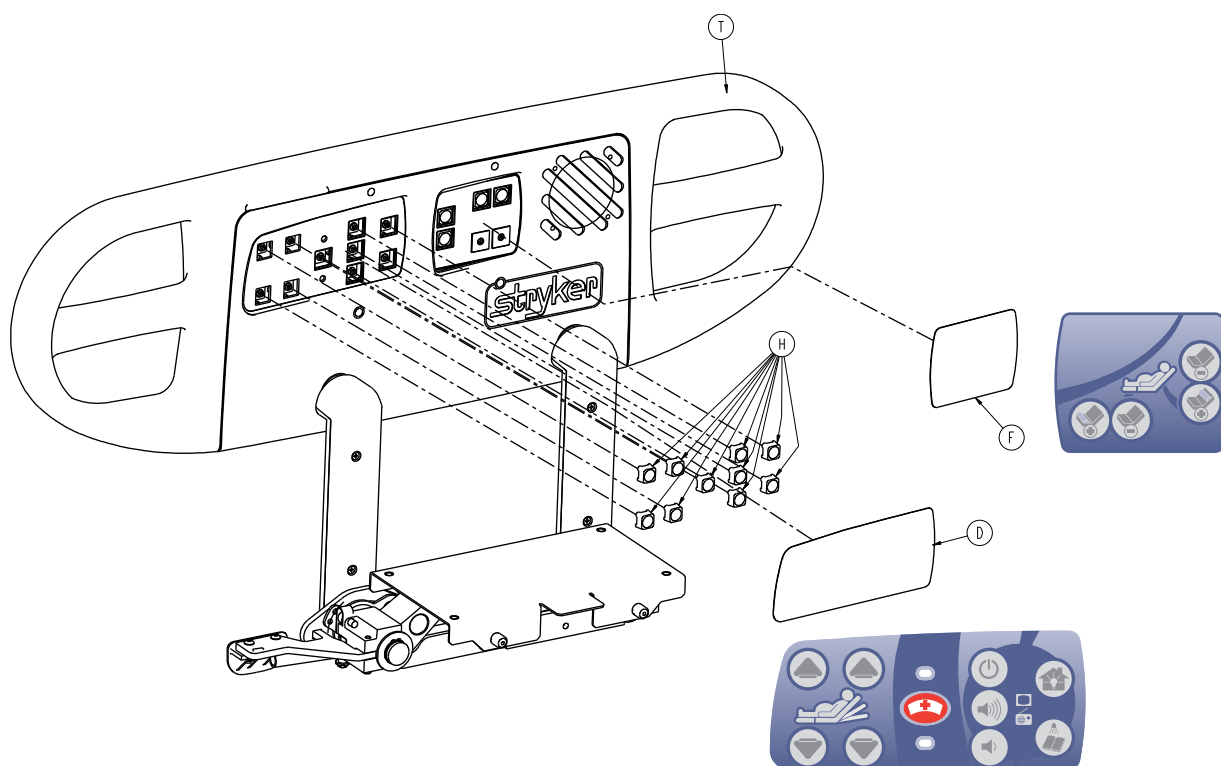

A	4700-020-009	2
B	0023-088-000	14
C	5000-080-016	2
D	4701-109-923	1
E	4701-109-924	1
F	4701-109-933	1
G	4701-109-934	1
H	3001-400-953	26
J	3001-400-522	2
K	3001-403-831	2
L	5000-220-008	1
M	5000-220-010	1
N	4701-109-913	1
P	4701-109-914	1
R	4701-120-095	1
T	4701-120-100	1

Rev A

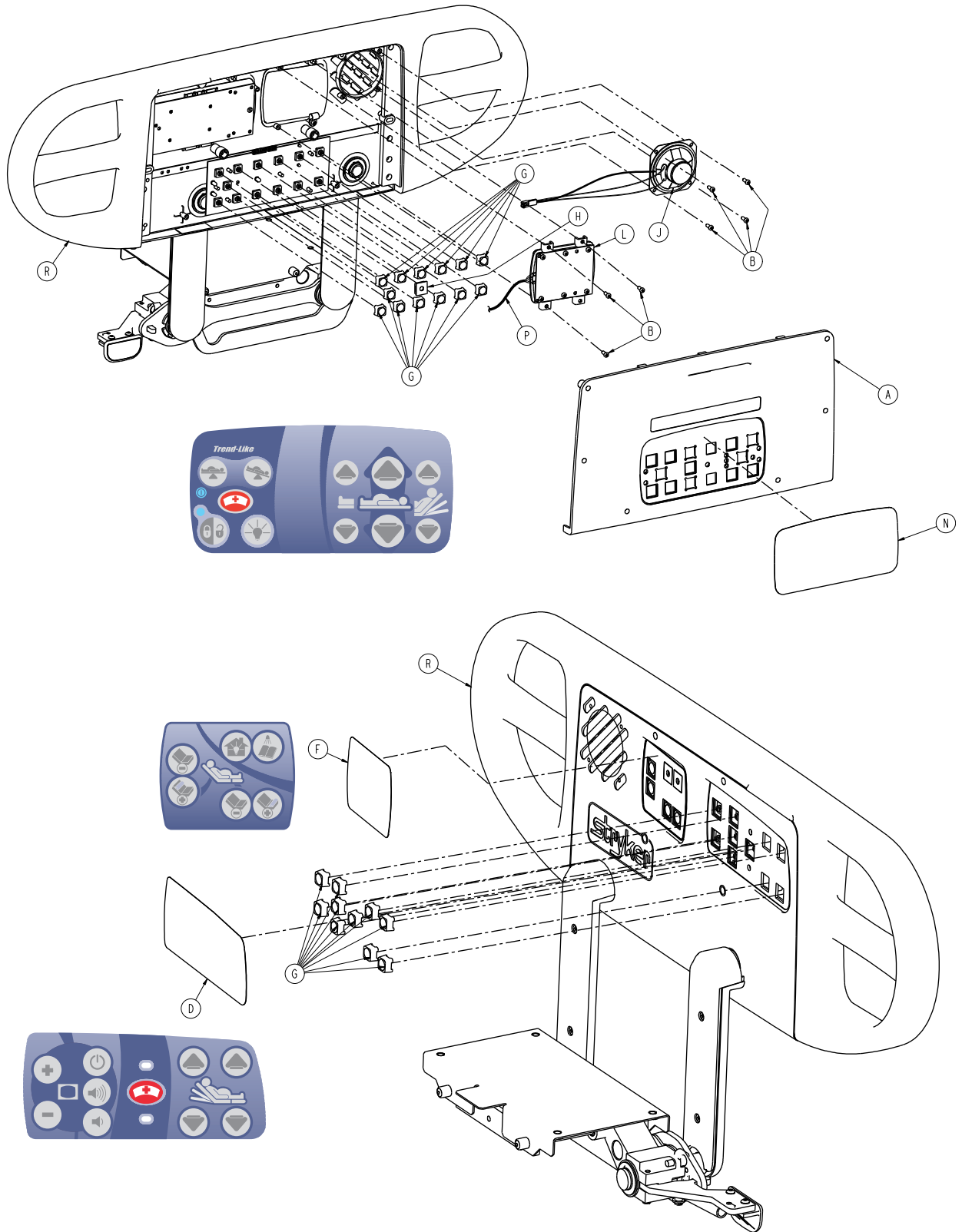


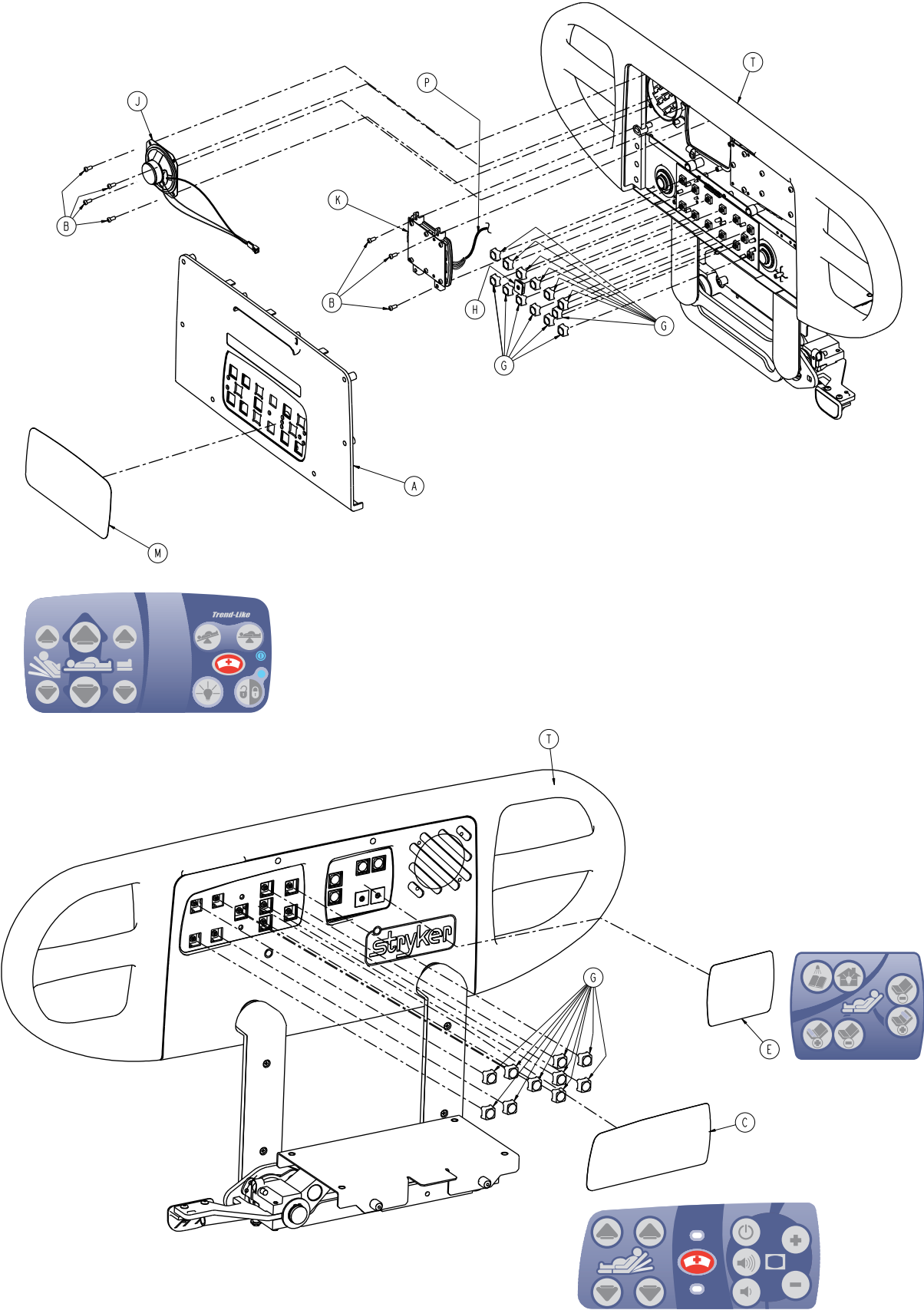



4701-120-030


A	4700-020-009	2
B	0023-088-000	14
C	4701-109-921	1
D	4701-109-922	1
E	4701-109-931	1
F	4701-109-932	1
G	3001-400-953	26
H	3001-400-522	2
J	3001-403-831	2
K	5000-020-026	1
L	5000-020-027	1
M	4701-109-913	1
N	4701-109-914	1
P	5000-080-016	2
R	4701-120-095	1
T	4701-120-100	1

Rev A

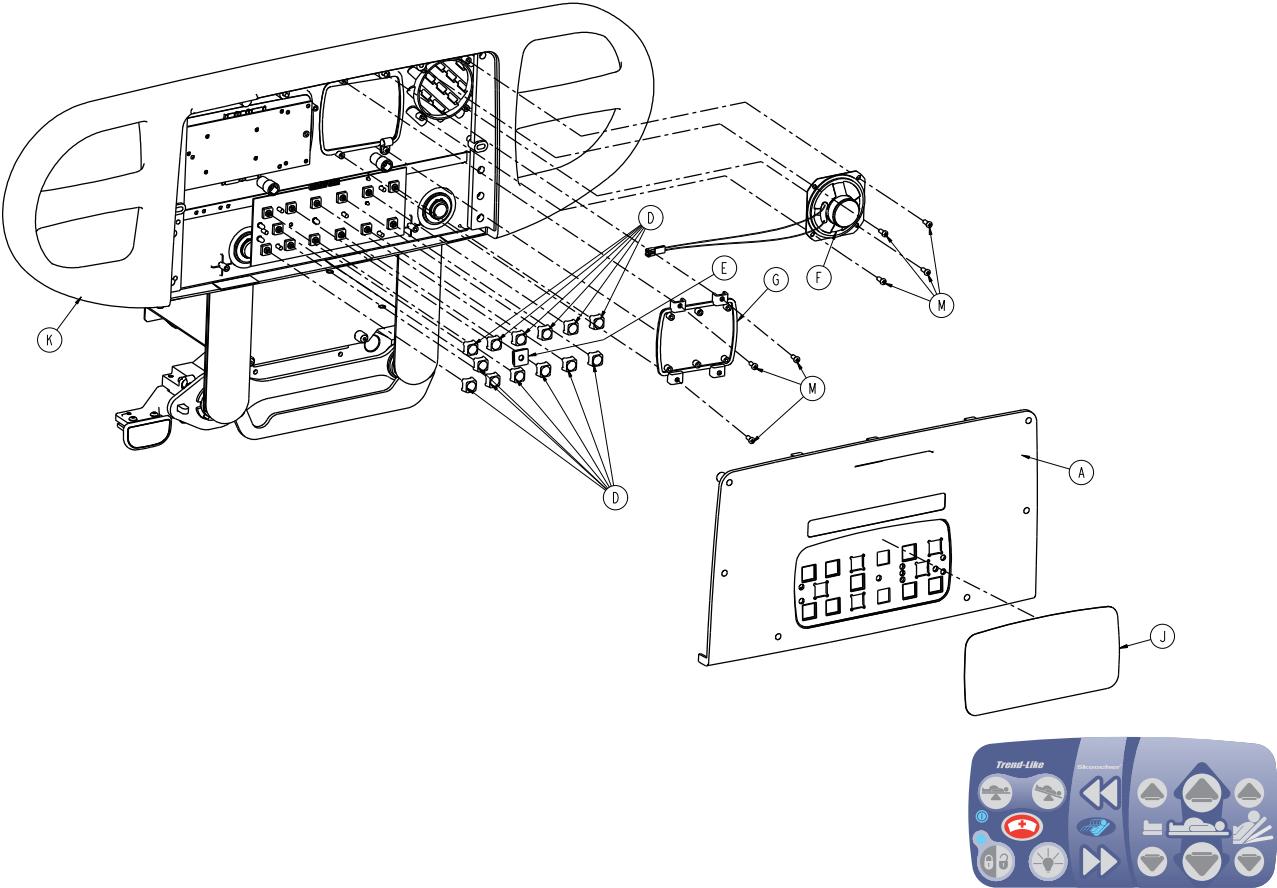


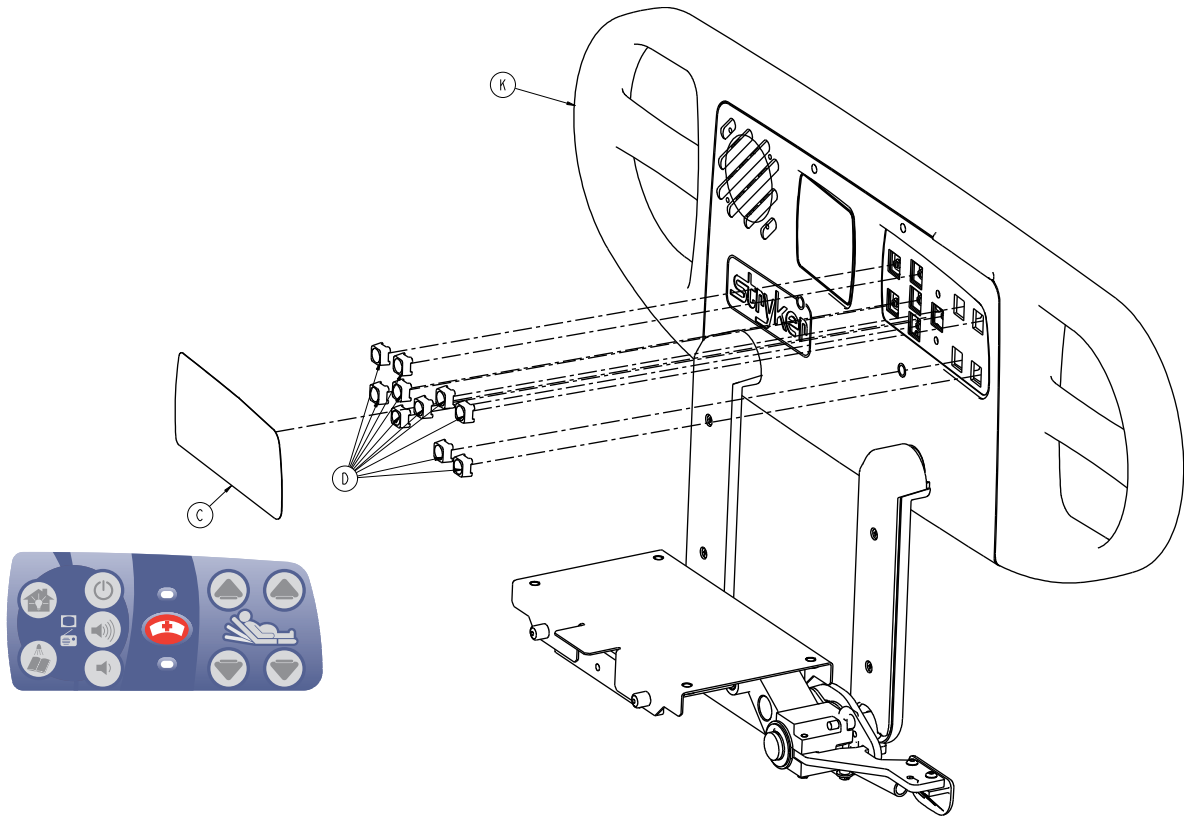

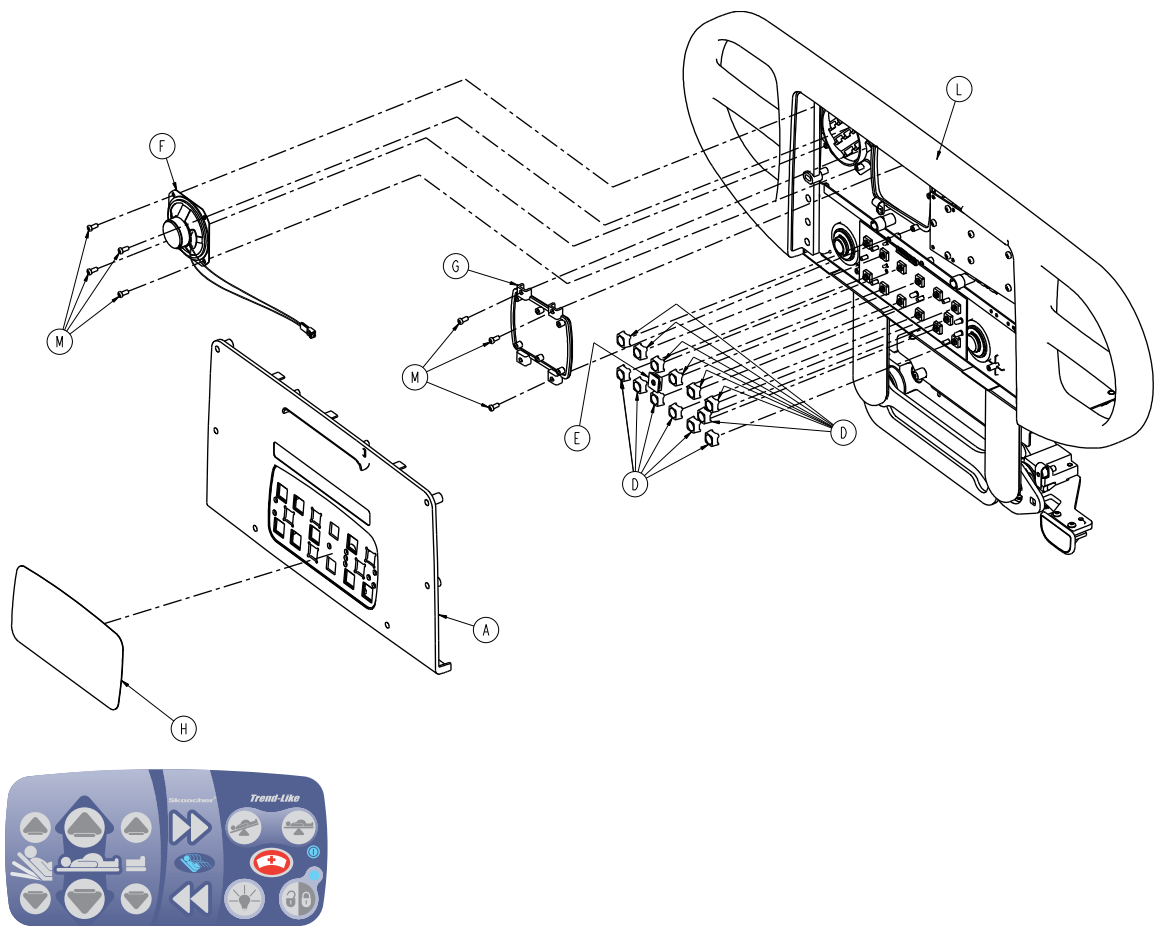

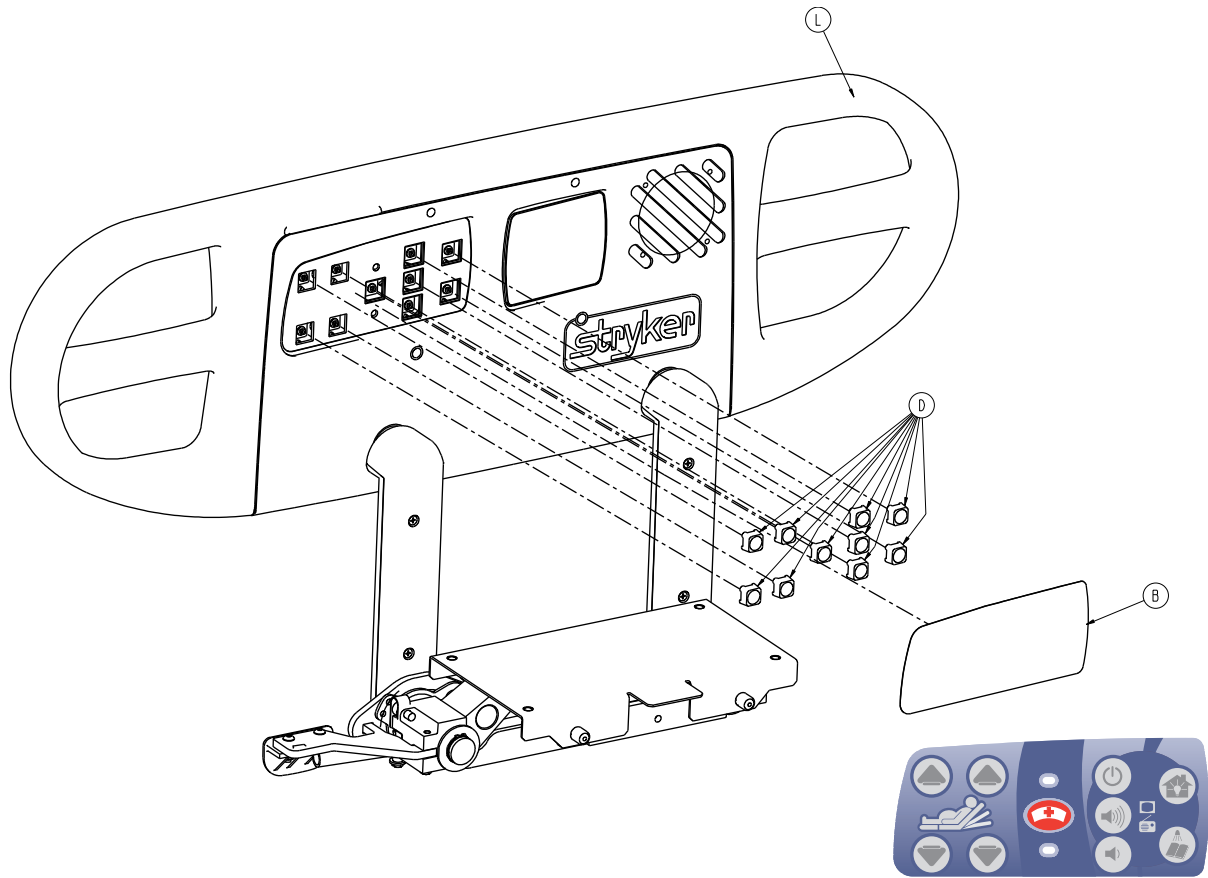


A	4700-020-009	2
B	4701-109-923	1
C	4701-109-924	1
D	3001-400-953	26
E	3001-400-522	2
F	3001-403-831	2
G	3001-400-535	2
H	4701-109-911	1
J	4701-109-912	1
K	4701-120-095	1
L	4701-120-100	1
M	0023-088-000	14



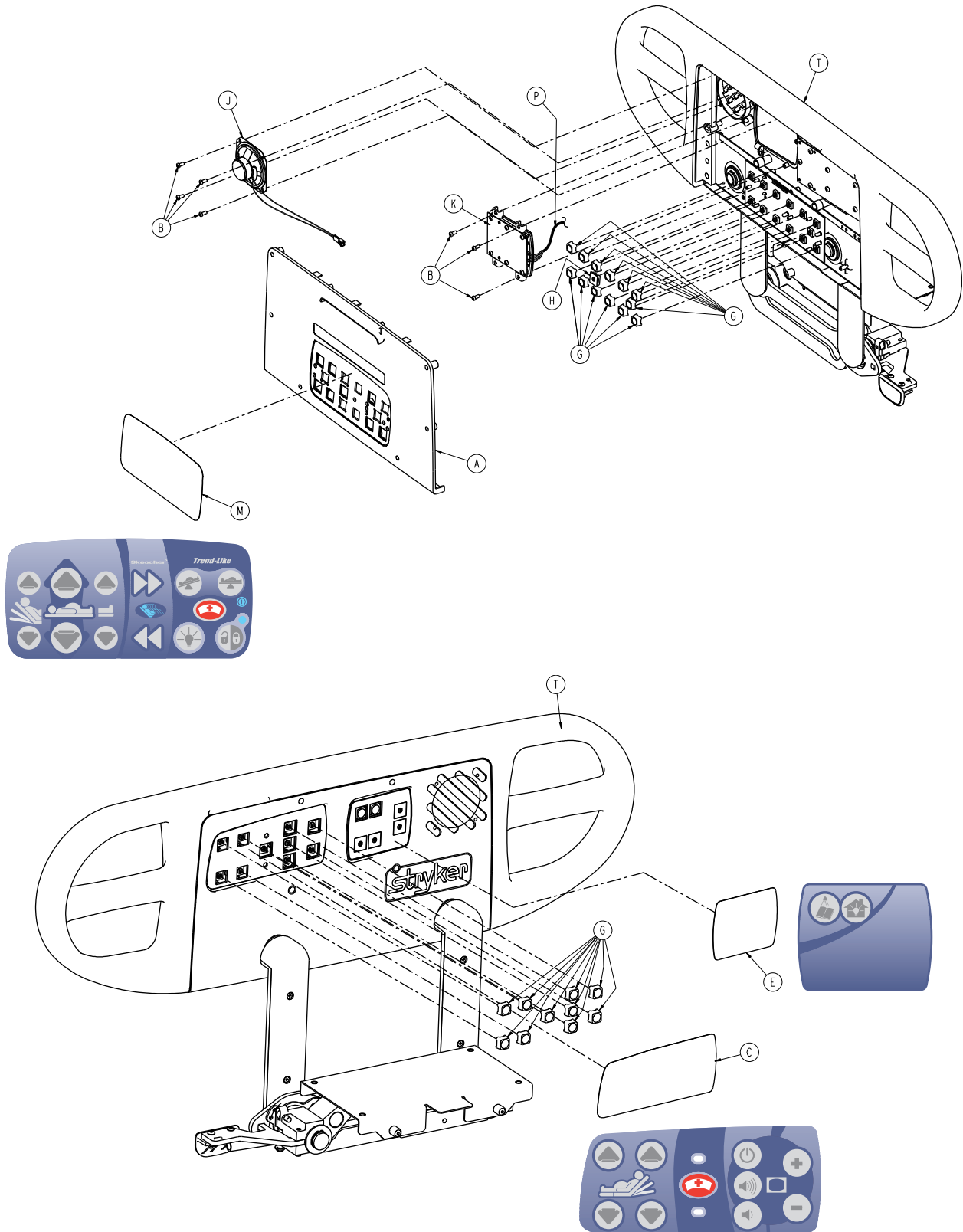


4701-120-040

A	4700-020-009	2
B	0023-088-000	14
C	4701-109-921	1
D	4701-109-922	1
E	4701-109-936	1
F	4701-109-937	1
G	3001-400-953	26
H	3001-400-522	2
J	3001-403-831	2
K	5000-020-028	1
L	5000-020-029	1
M	4701-109-911	1
N	4701-109-912	1
P	5000-080-016	2
R	4701-120-095	1
T	4701-120-100	1

Rev A

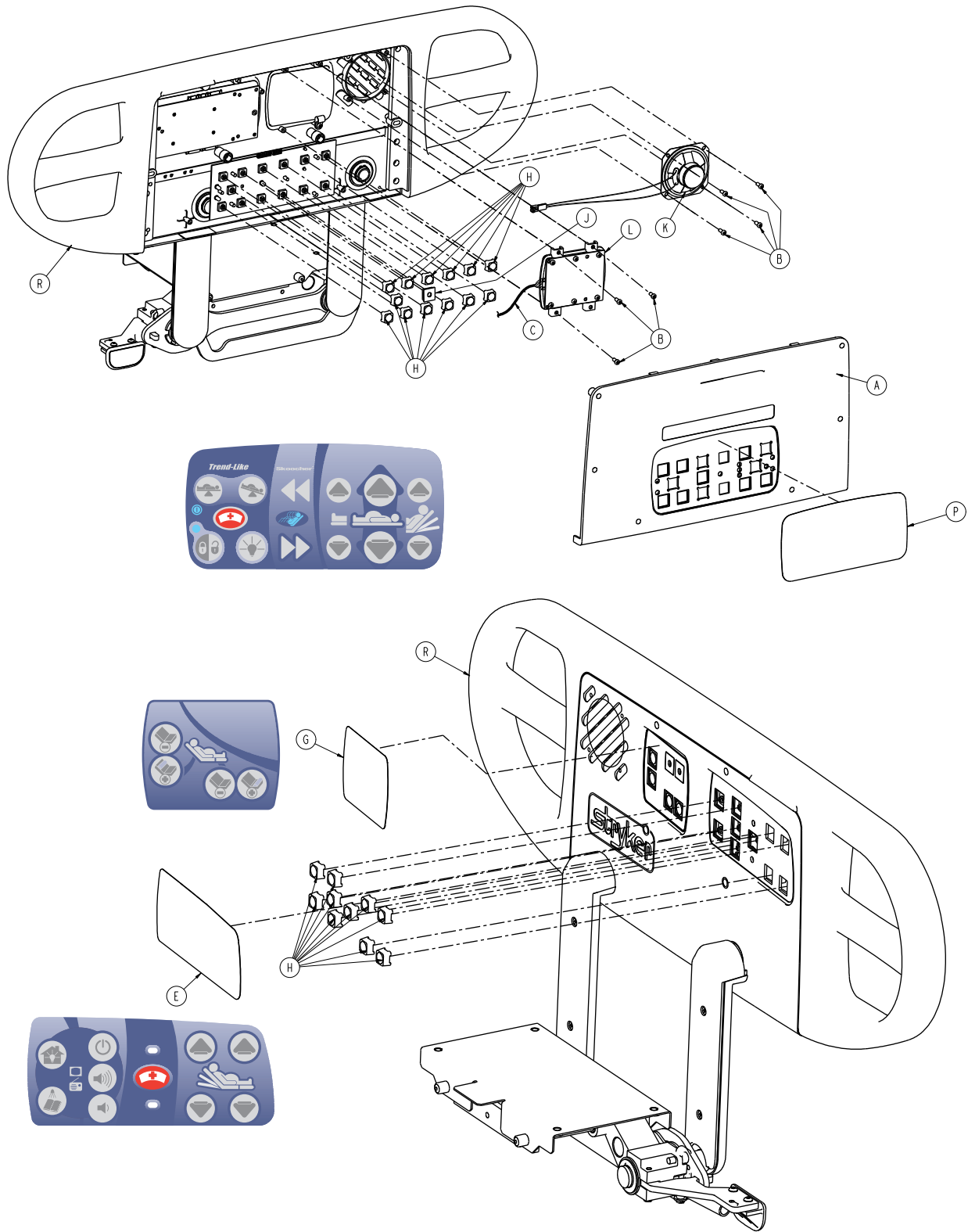


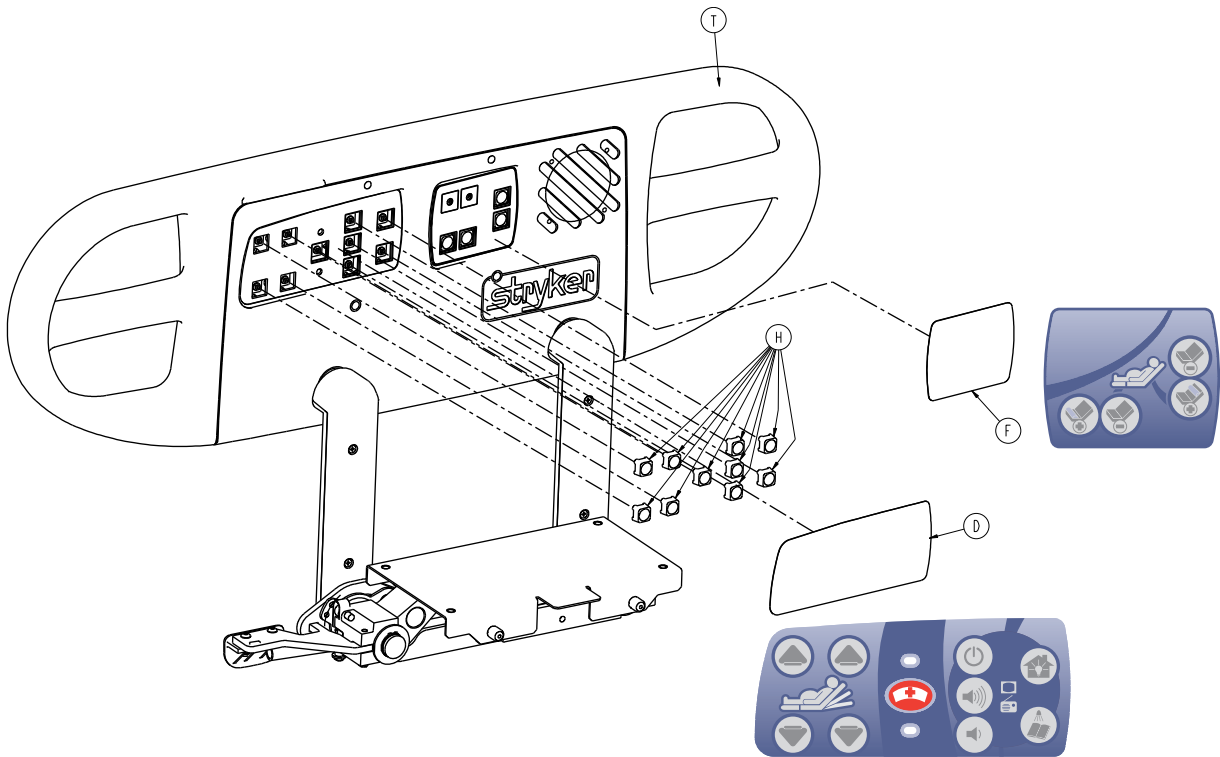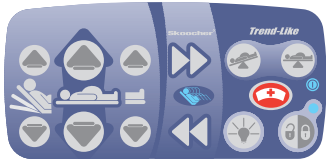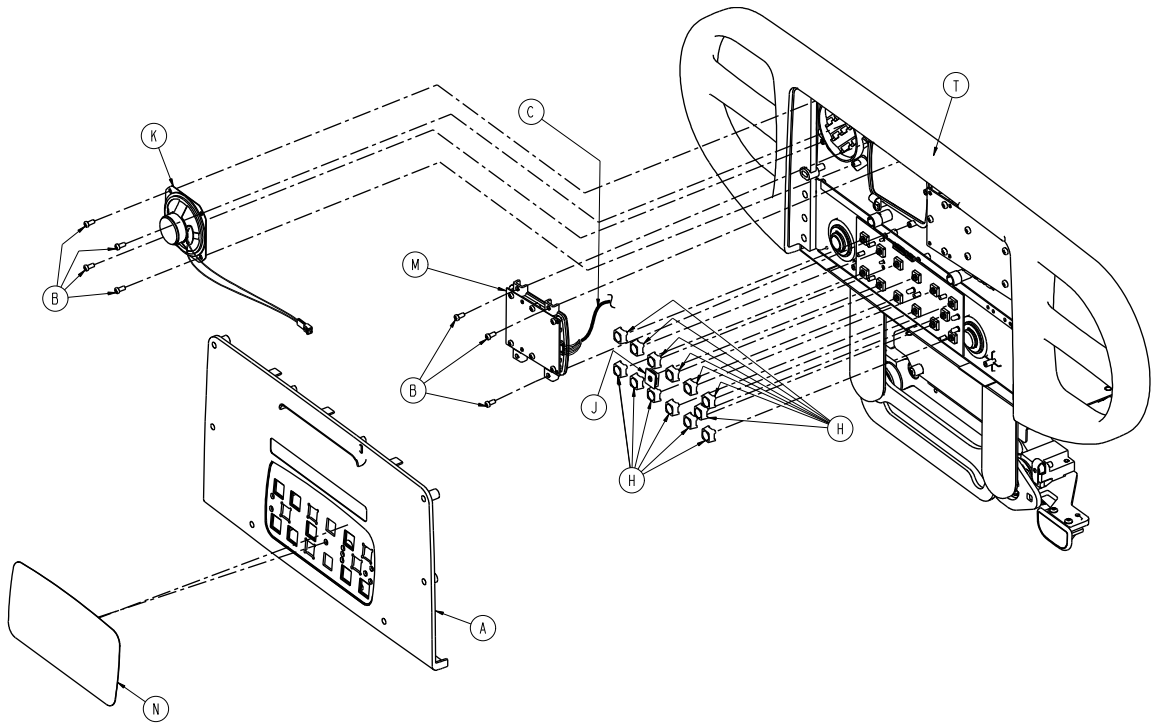



4701-120-015

A	4700-020-009	2
B	0023-088-000	14
C	5000-080-016	2
D	4701-109-923	1
E	4701-109-924	1
F	4701-109-933	1
G	4701-109-934	1
H	3001-400-953	26
J	3001-400-522	2
K	3001-403-831	2
L	5000-220-008	1
M	5000-220-010	1
N	4701-109-911	1
P	4701-109-912	1
R	4701-120-095	1
T	4701-120-100	1

Rev A

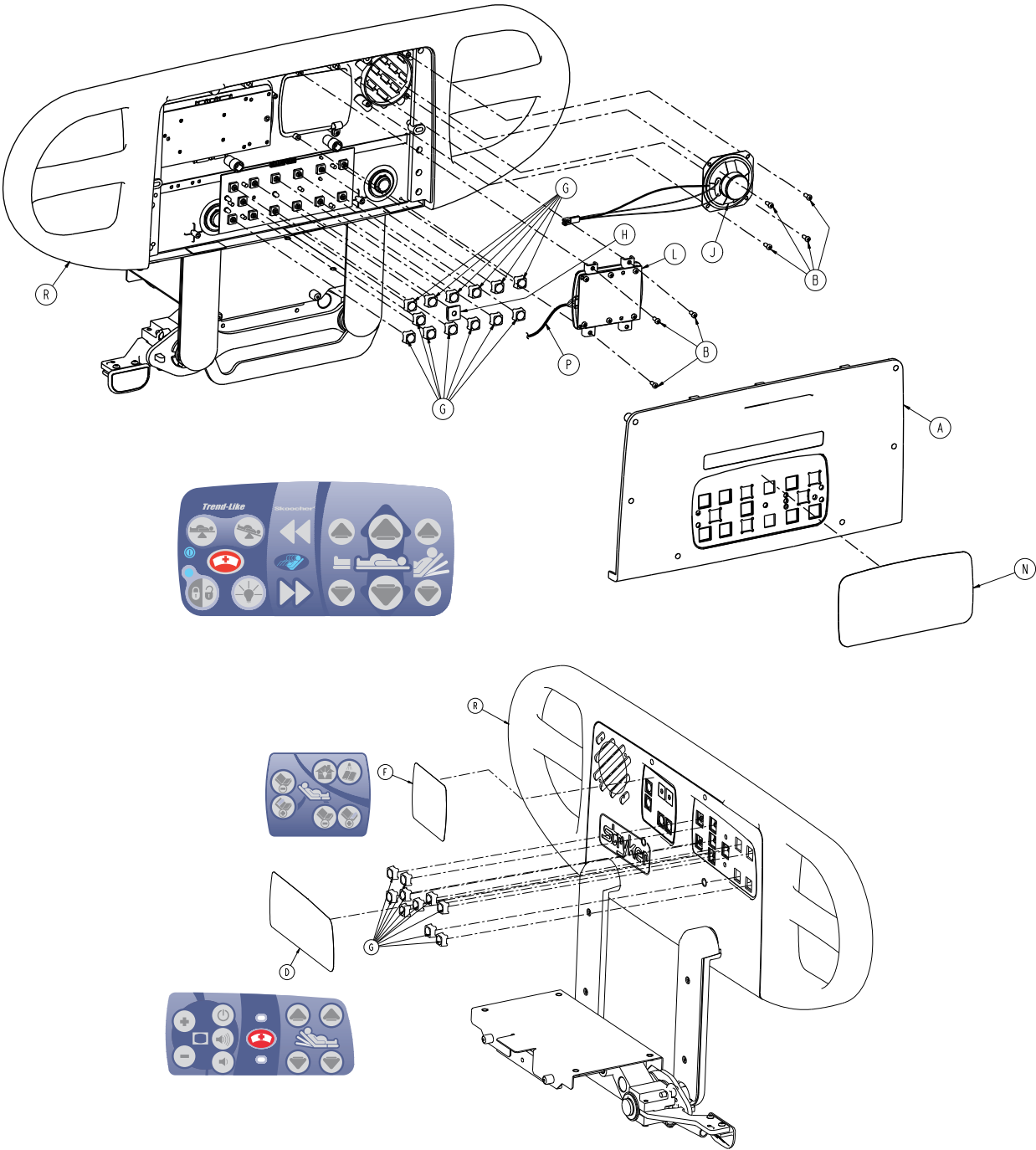


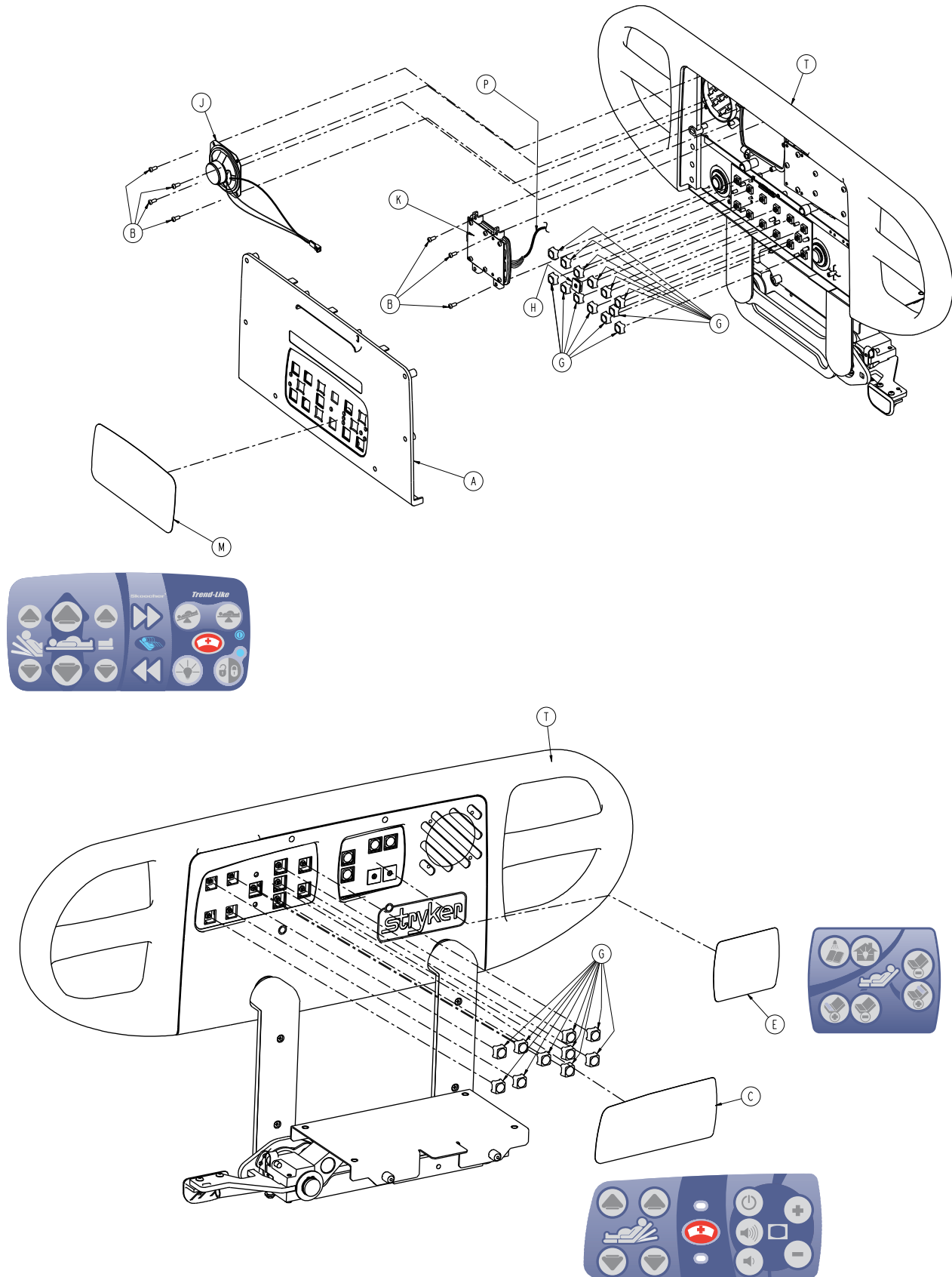

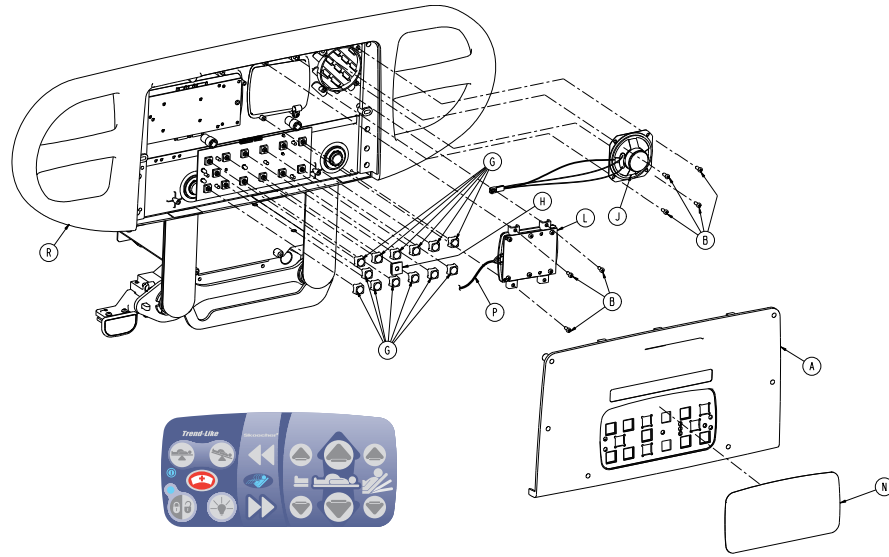


A	4700-020-009	2
B	0023-088-000	14
C	4701-109-921	1
D	4701-109-922	1
E	4701-109-931	1
F	4701-109-932	1
G	3001-400-953	26
H	3001-400-522	2
J	3001-403-831	2
K	5000-020-026	1
L	5000-020-027	1
M	4701-109-911	1
N	4701-109-912	1
P	5000-080-016	2
R	4701-120-095	1
T	4701-120-100	1

Rev B

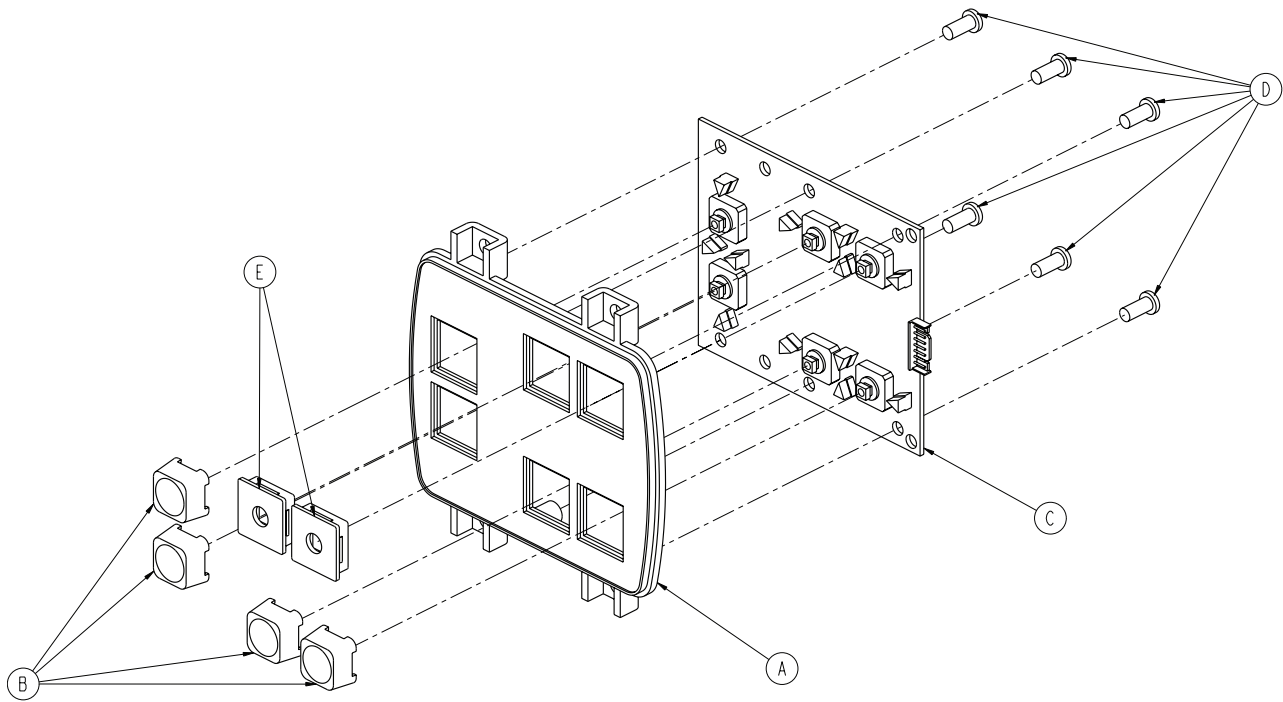


A	3001-400-524	1
B	3001-400-953	6
C	5000-400-930	1
D	0023-088-000	6
E	3001-400-522	4

Rev A

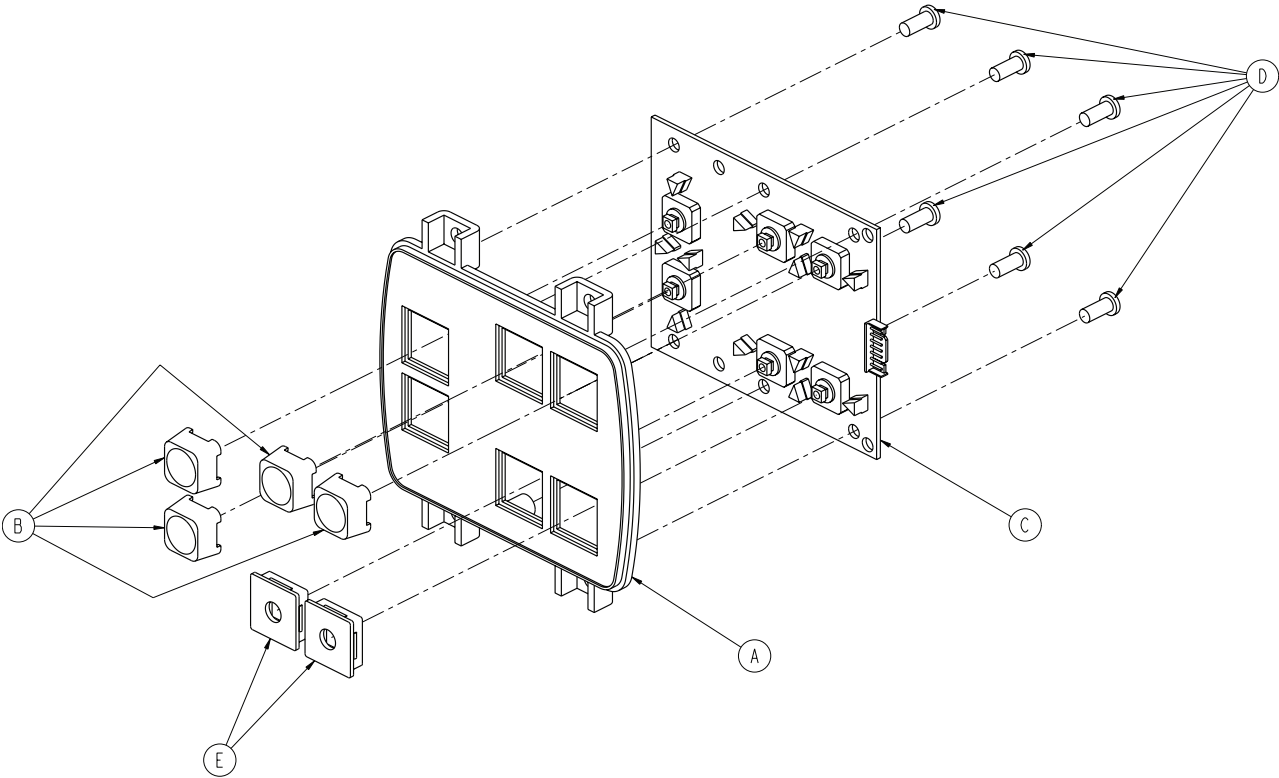


A	3001-400-524	1
B	3001-400-953	6
C	5000-400-920	1
D	0023-088-000	6
E	3001-400-522	4

Rev A

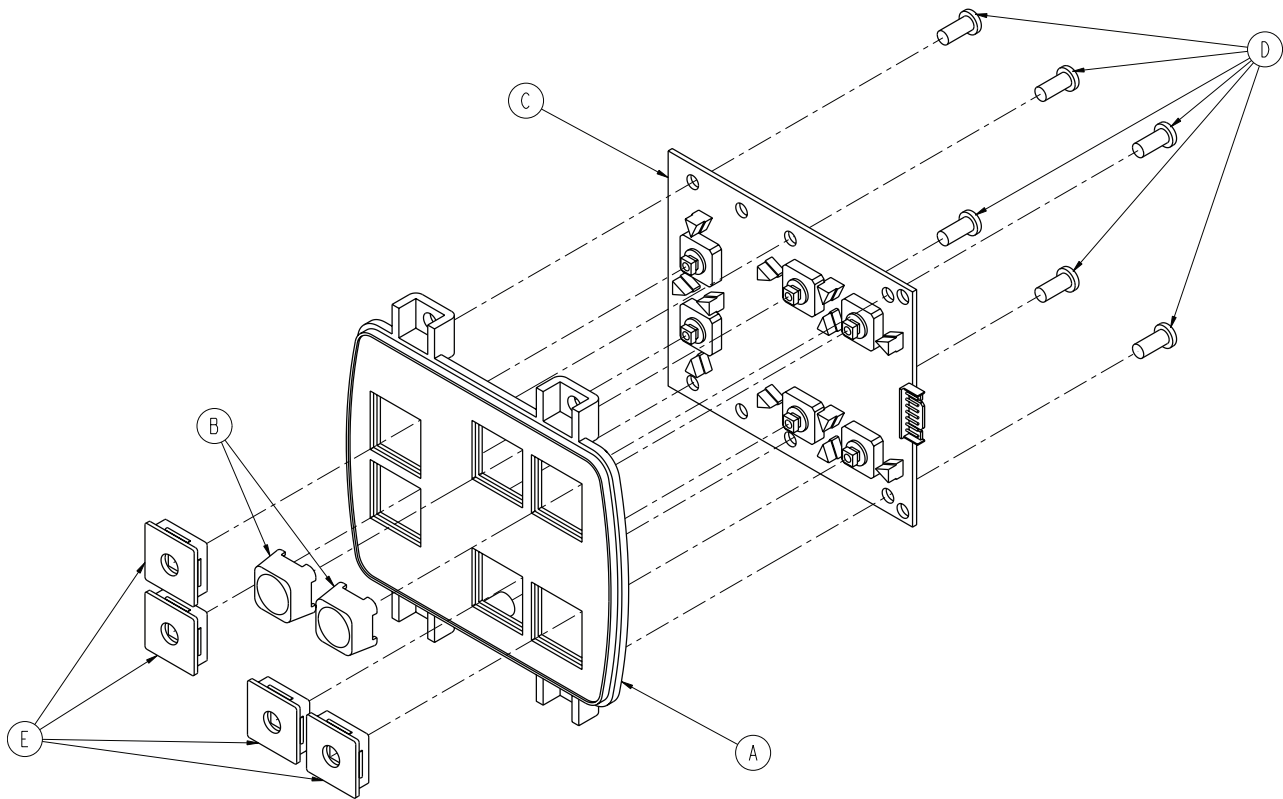


A	3001-400-524	1
B	3001-400-953	6
C	5000-400-930	1
D	0023-088-000	6
E	3001-400-522	4

Rev A

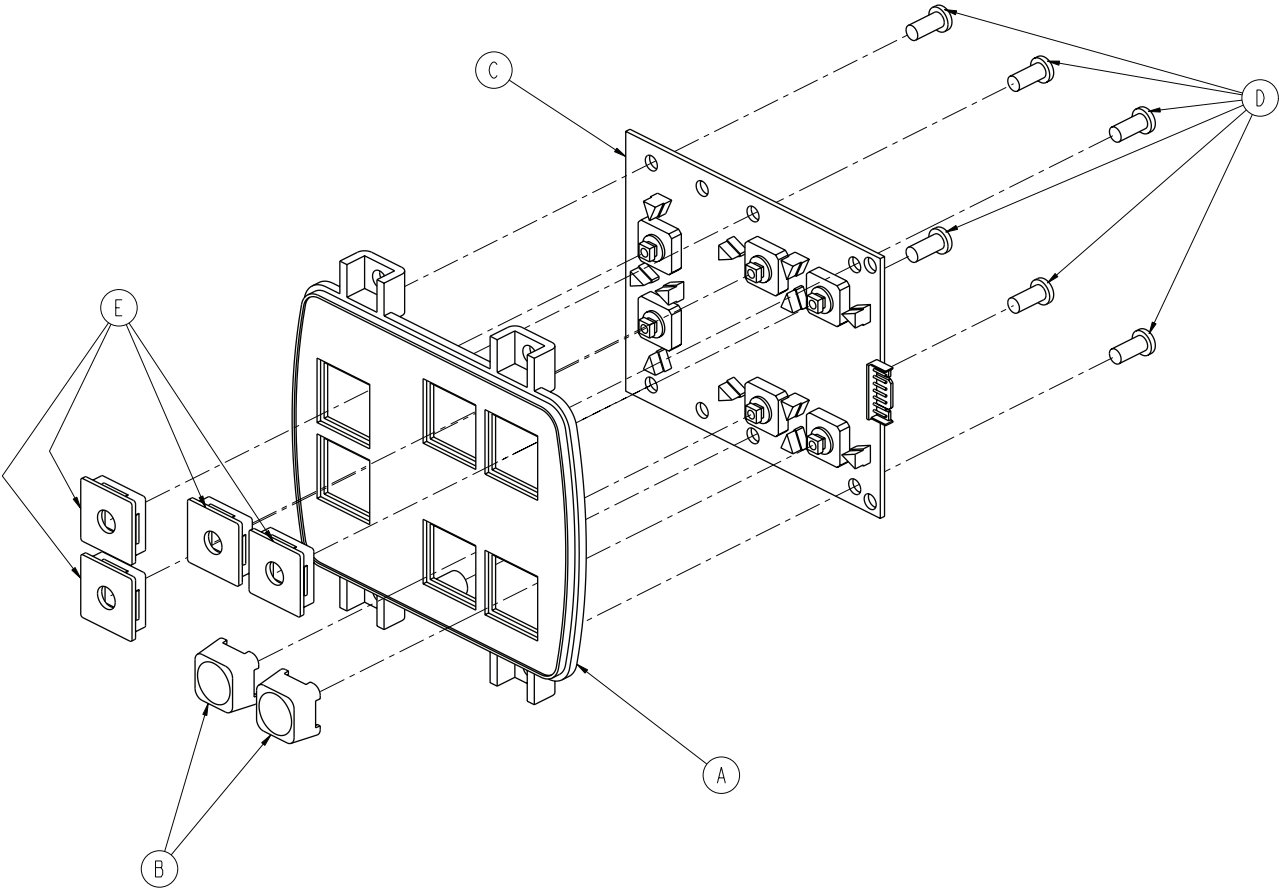


A	3001-400-524	1
B	3001-400-953	6
C	5000-400-920	1
D	0023-088-000	6
E	3001-400-522	4

Rev A

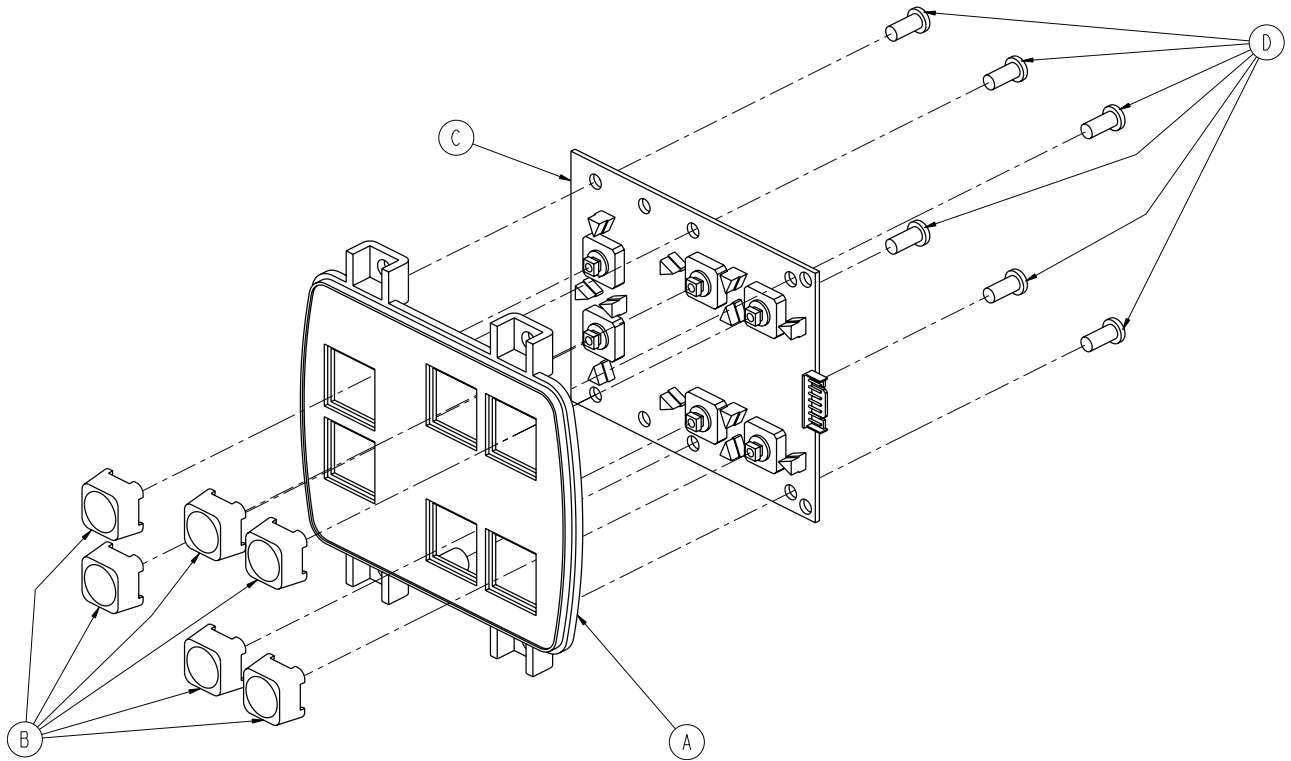


A	3001-400-524	1
B	3001-400-953	6
C	5000-400-930	1
D	0023-088-000	6


Rev A

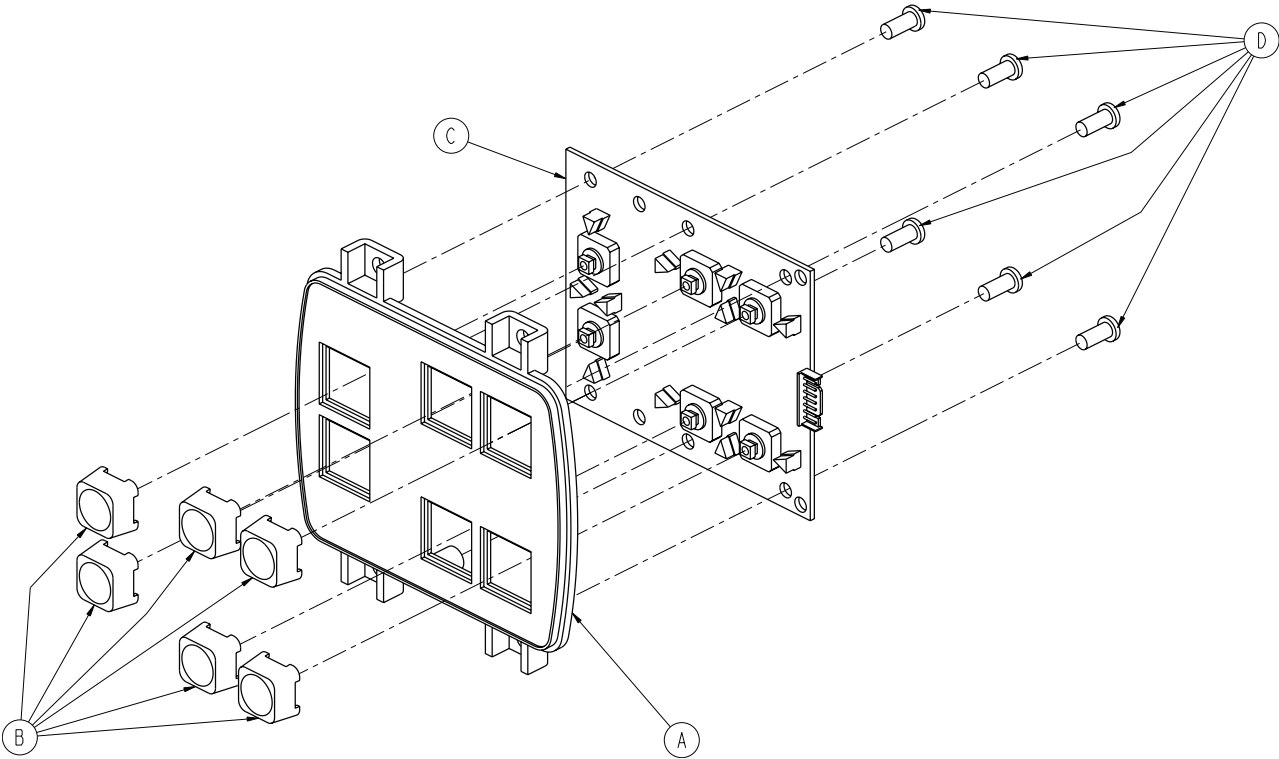


A	3001-400-524	1
B	3001-400-953	6
C	5000-400-920	1
D	0023-088-000	6

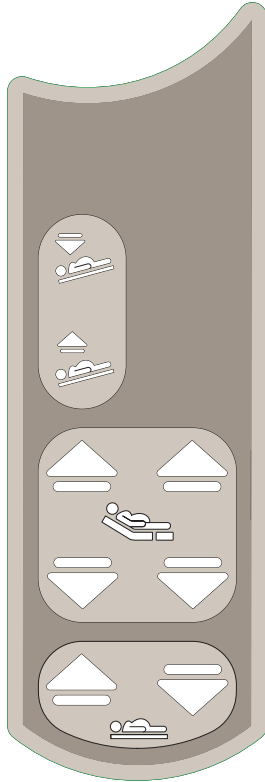

5000-081-021 Rev F
5000-090-062 Rev E

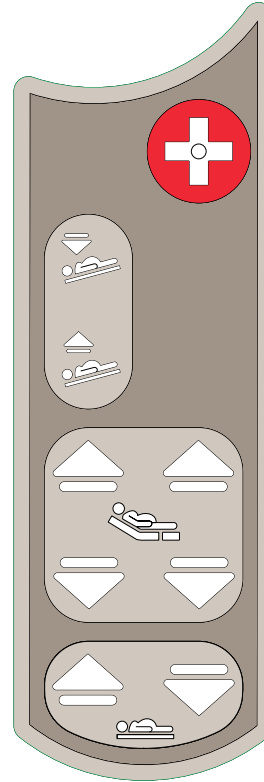

5000-081-023 Rev C
5000-090-056 Rev A

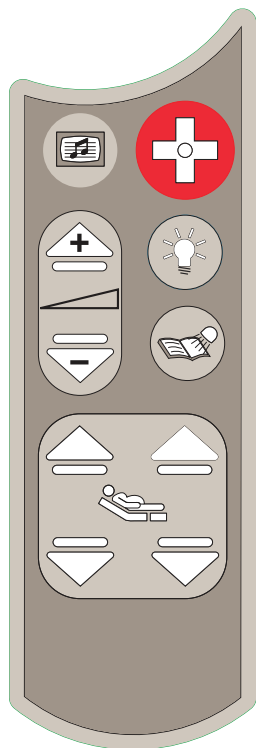


5000-081-020 Rev F

5000-090-055 Rev D

4701-130-125

Rev A

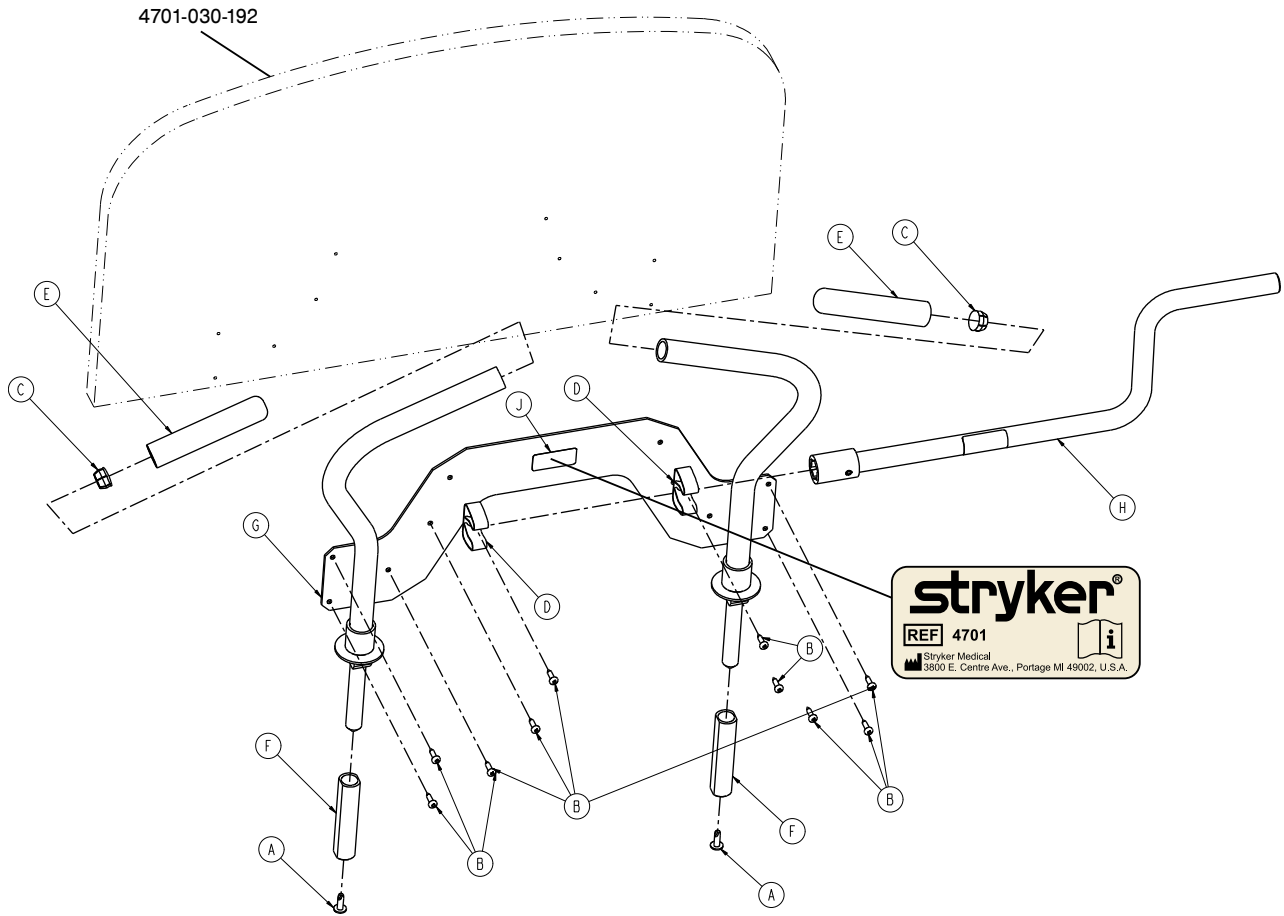


A	0007-065-000	2
B	0023-311-000	10
C	0037-062-000	2
D	0058-065-000	2
E	4700-030-057	2
F	4701-030-187	2
G	4701-030-195	1
H	4701-136-020	1
J	4701-109-981	1

4701-030-192 Rev F


4701-030-191	4701-030-203
4701-030-193	4701-030-204
4701-030-194	4701-030-206
4701-030-198	

Rev A

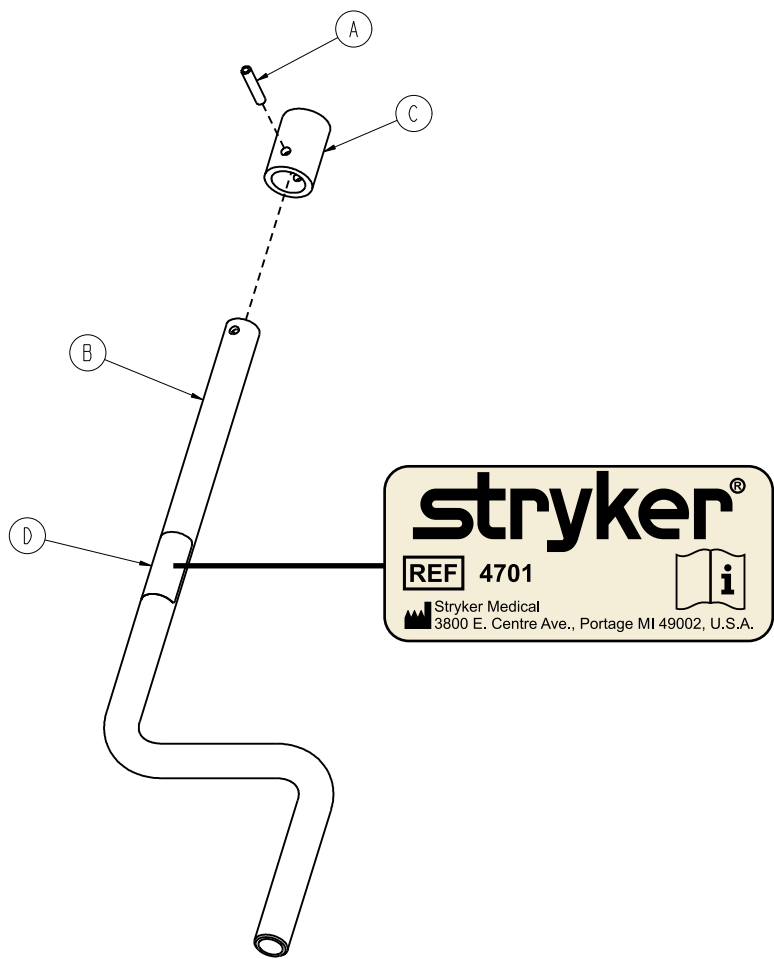

A	0026-267-000	1
B	4701-036-018	1
C	4701-036-019	1
D	4701-109-981	1


Rev A

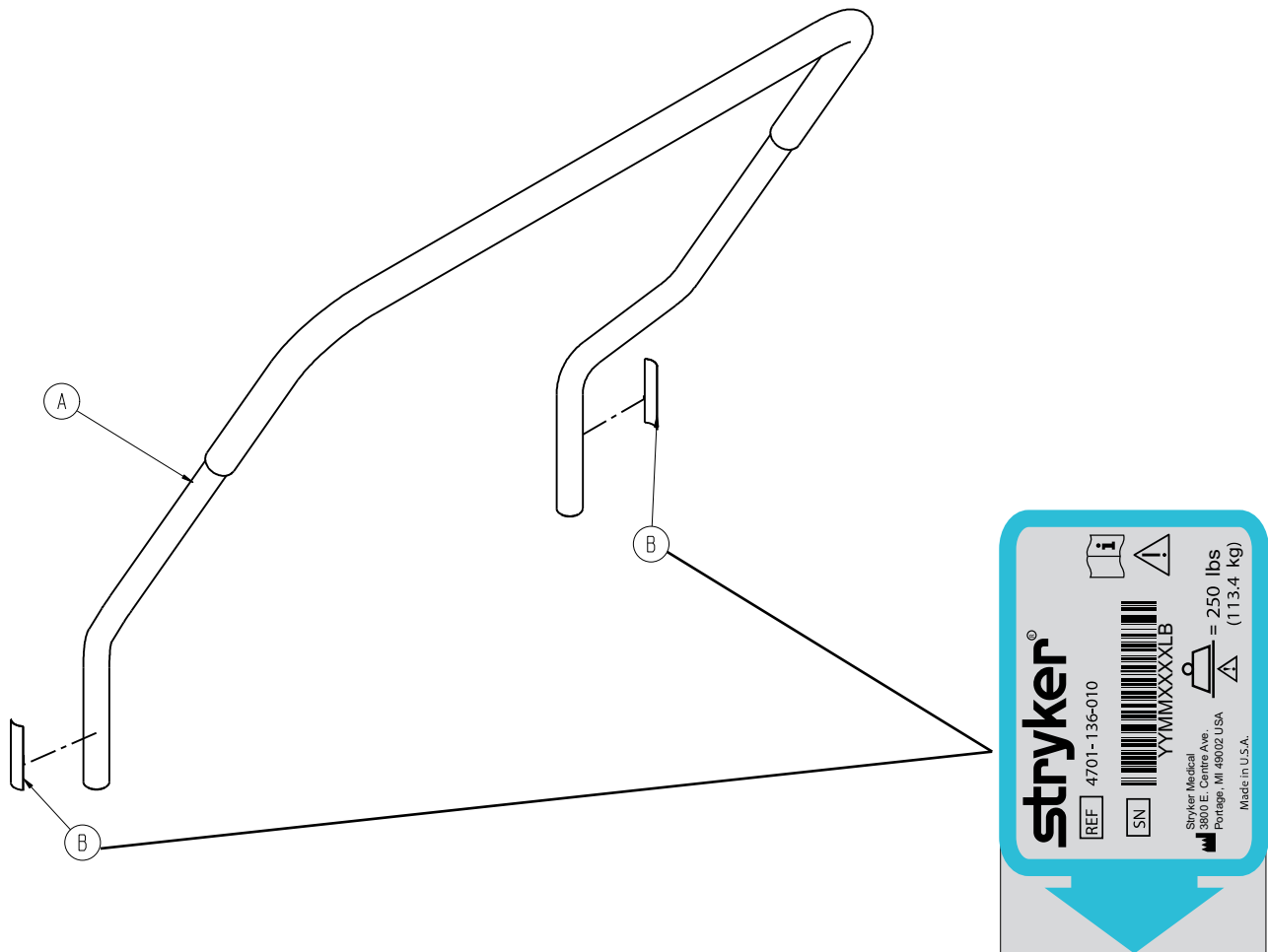

A	4701-036-014
B	4701-109-966


1
2

Rev B

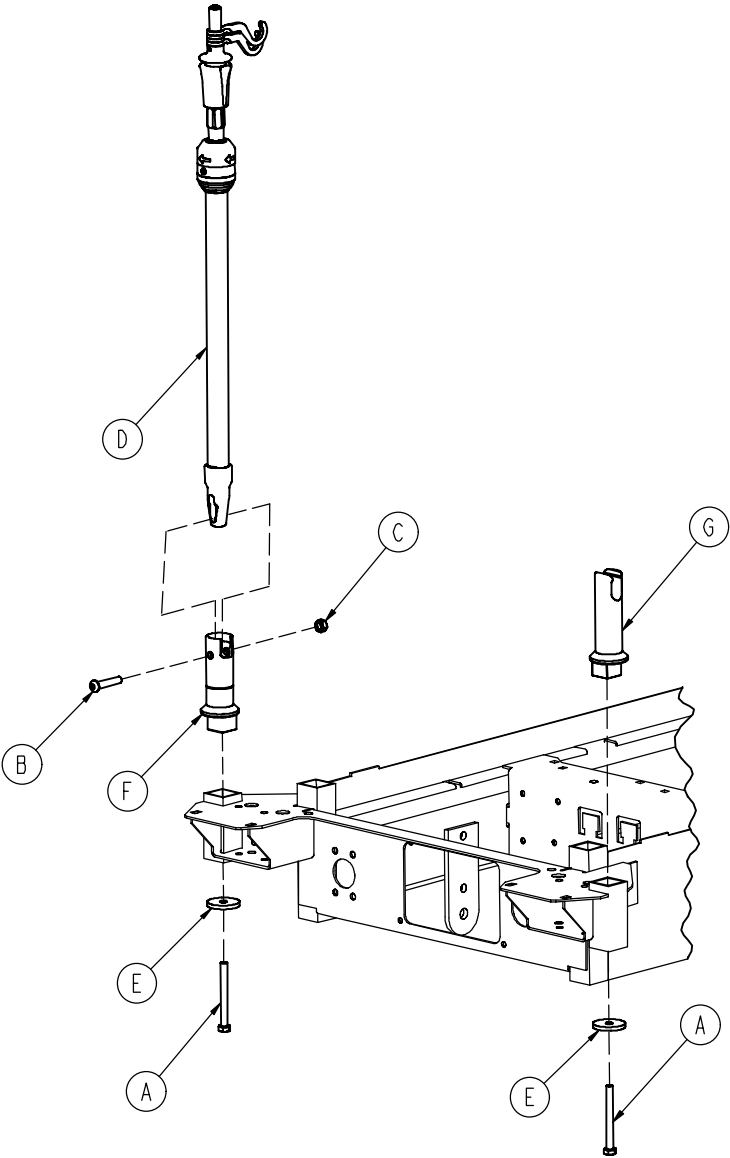


A	0003-383-000	2
B	0004-199-000	1
C	0016-116-000	1
D	1211-211-010	1
E	4701-036-023	2
F	4701-036-028	1
G	4701-036-027	1

Rev V

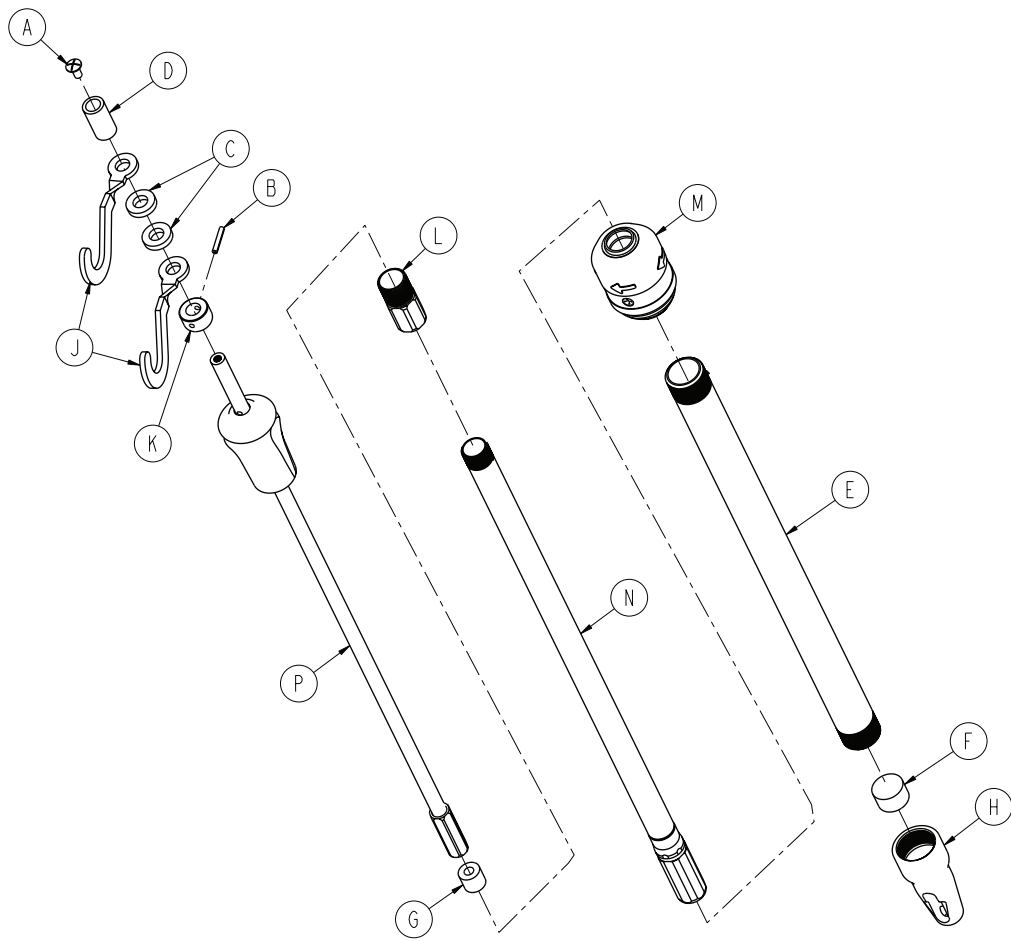

A	0007-004-000	1
B	0026-076-000	1
C	0052-017-000	2
D	0926-400-162	1
E	1001-161-023	1
F	1001-259-013	1
G	1001-359-014	1
H	1001-359-112	1
J	1010-059-016	2
K	1010-061-014	1
L	1211-110-016	1
M	1211-210-026	1
N	1211-210-031	1
P	1211-210-032	1

Rev A

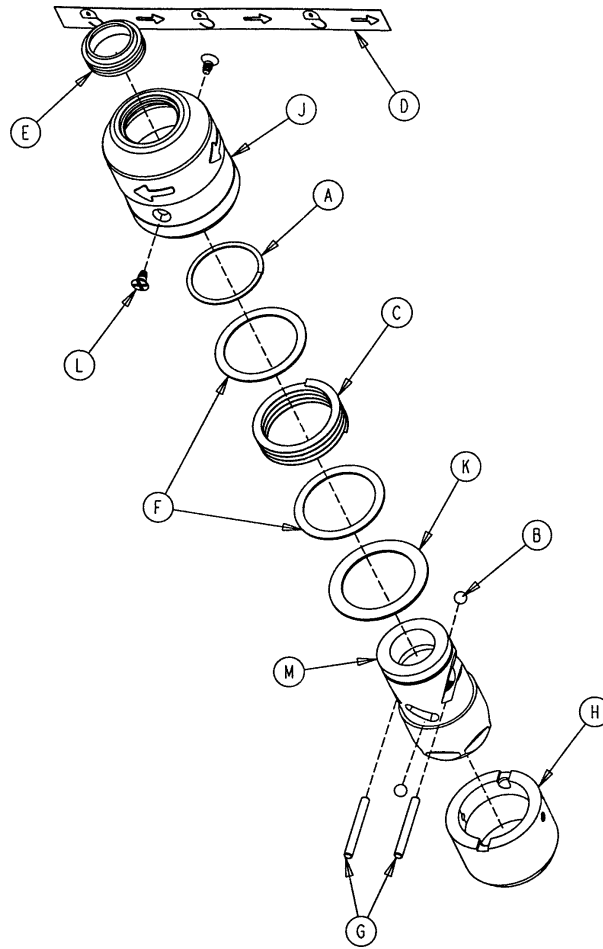


A	0028-167-000	1
B	0031-004-000	2
C	0038-392-000	1
D	1211-091-034	1
E	1211-110-018	1
F	1211-110-020	2
G	1211-110-021	2
H	1211-110-022	1
J	1211-110-024	1
K	1211-110-035	1
L	1211-110-036	2
M	1211-210-023	1

Recycling Passports



Assembly Part Number: 4701-080-030 (Reference only)

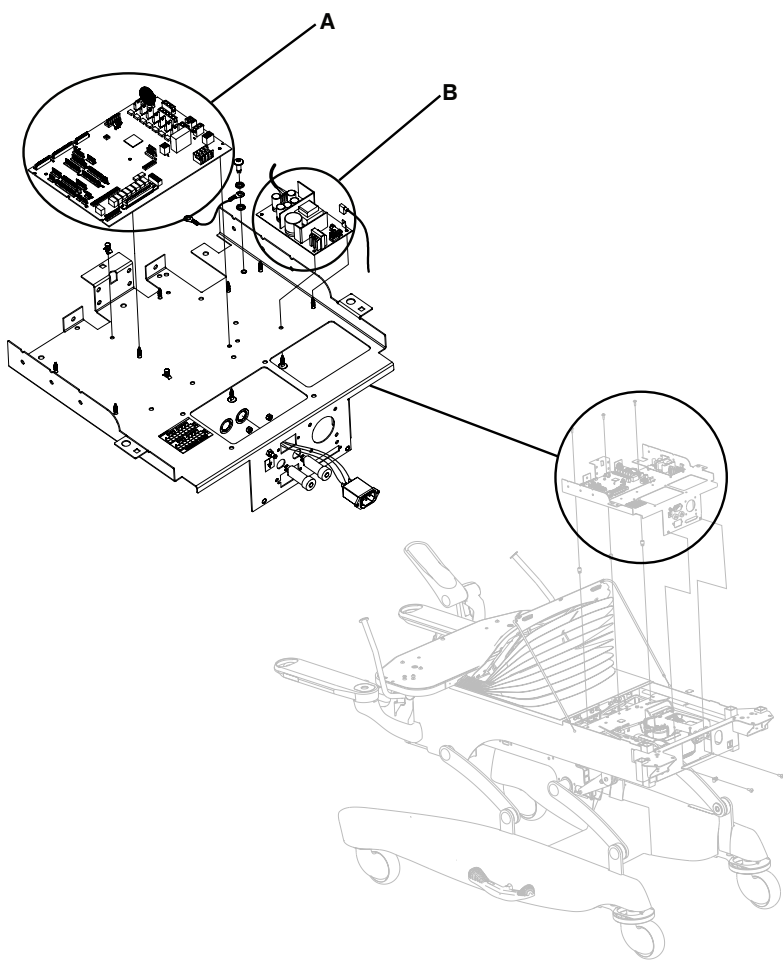


Item	Recycling/Material code	Important information	Qty
A	CPU Board (3002-407-950)		1
B	Power Supply (0059-157-000)		1

Recycling Passports



Assembly Part Number: 4701-080-055 (Reference only)

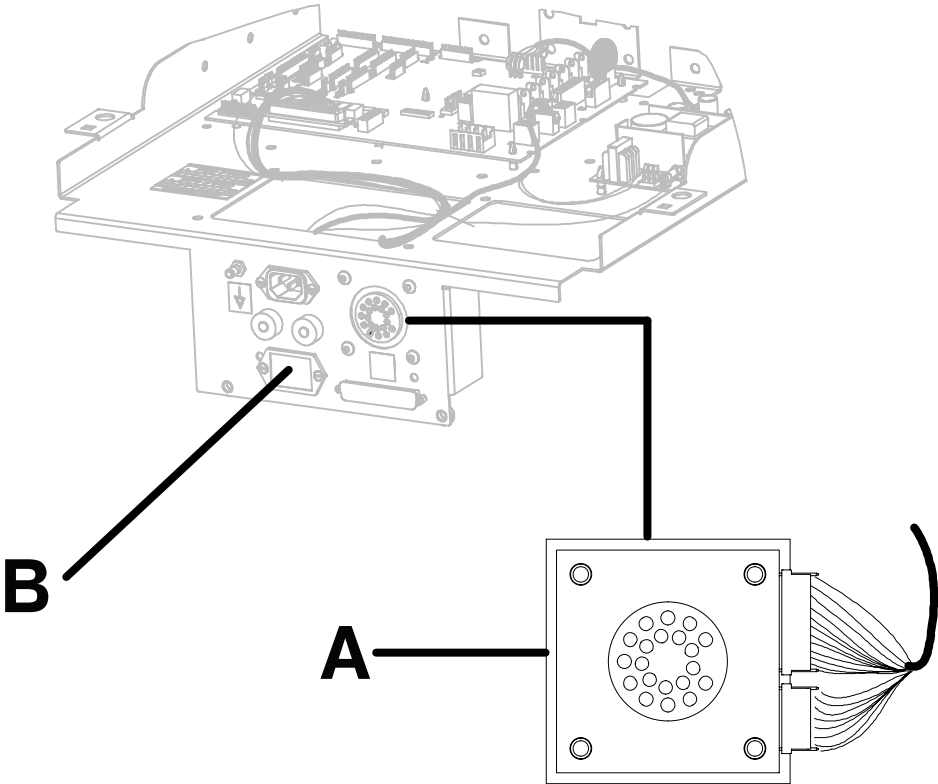


Item	Recycling/Material code	Important information	Qty
A	Pendant Port PCB (3001-314-920)		1
B	9V Battery (5010-080-020)		1

Recycling Passports



Assembly Part Number: 4701-045-100 (Reference only)

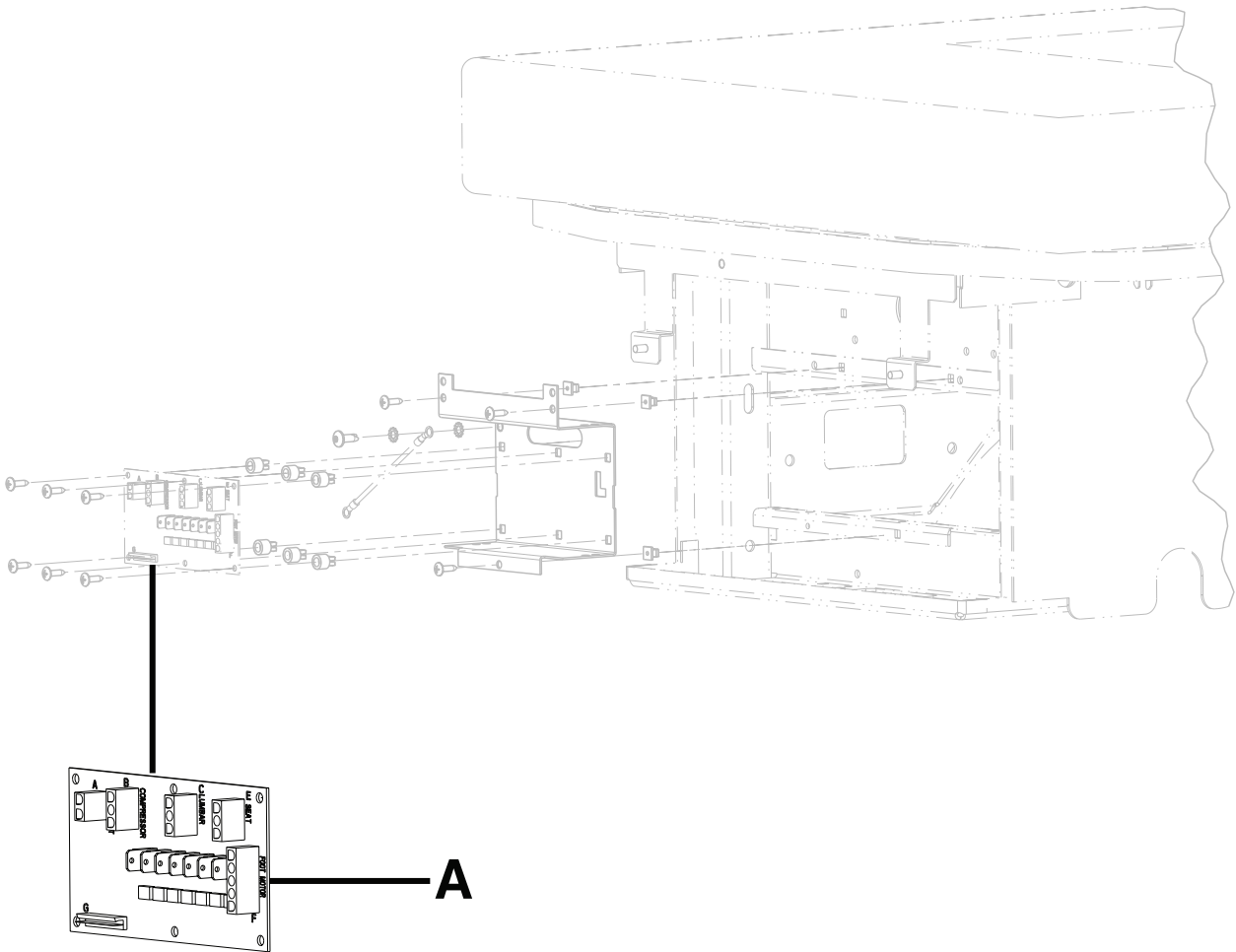


Item	Recycling/Material code	Important information	Qty
A	Foot Power Board (4701-080-050)		1

Recycling Passports



Assembly Part Number:
4701-120-090
4701-120-085 (Reference Only)

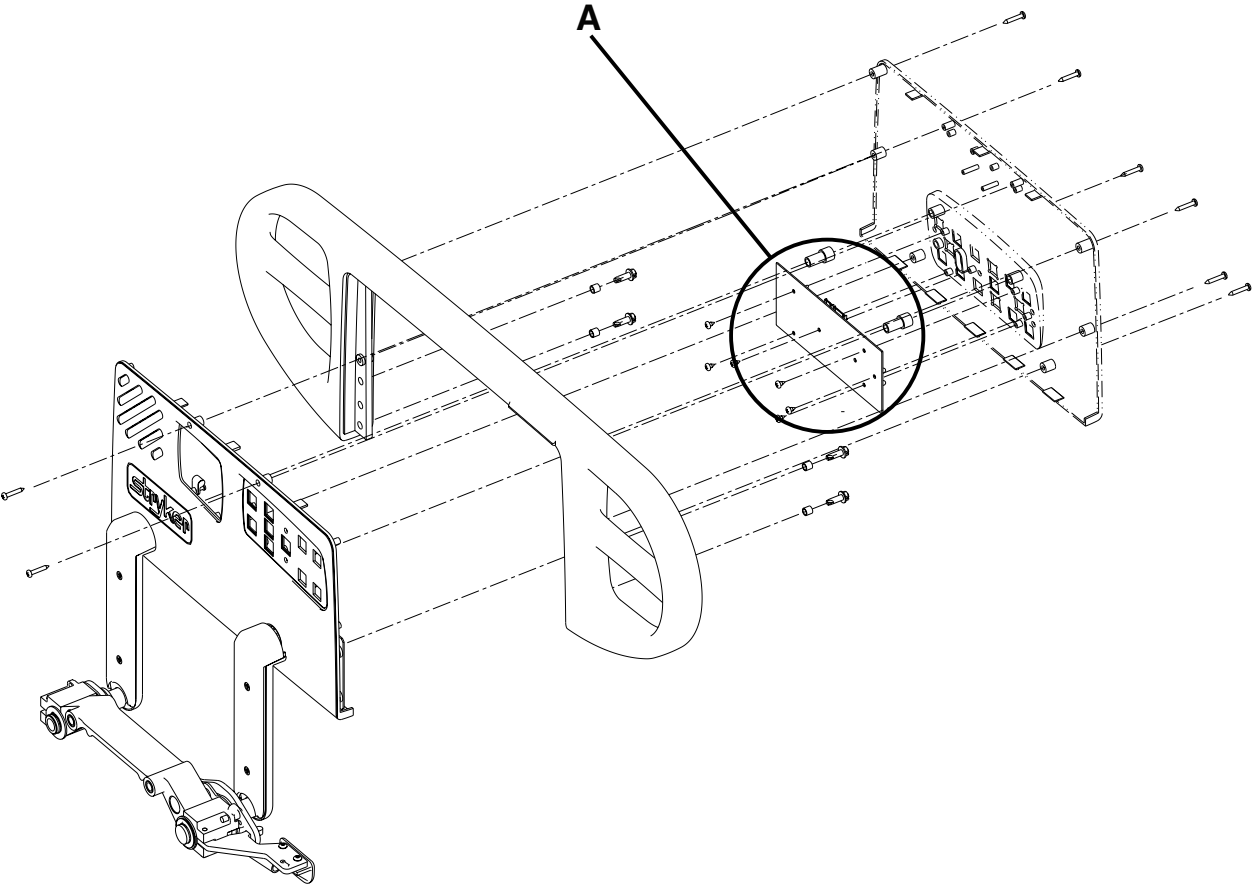


Item	Recycling/Material code	Important information	Qty
A	Siderail PCB, Outer (4701-080-060 Right) and (4701-080-070 Left)		1

Recycling Passports



Assembly Part Number:
4701-120-025
4701-120-030
4701-120-035
4701-120-040
(Reference Only)

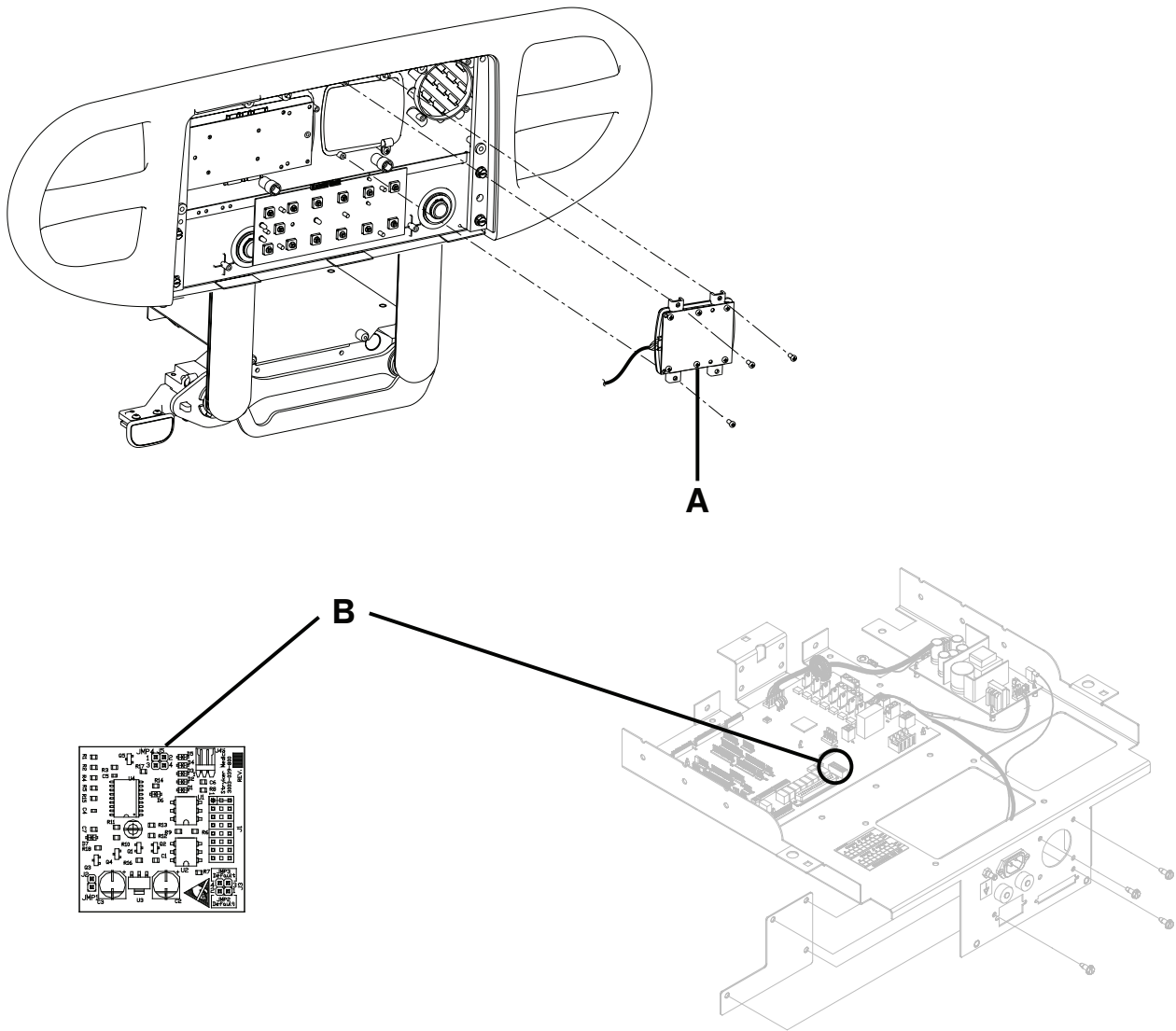


Item	Recycling/Material code	Important information	Qty
A	TV Module, Left (5000-020-028) TV Module, Right (5000-020-029)		1
B	Smart TV PCB (3003-039-800)		1

Recycling Passports



Assembly Part Number:
5000-220-008
5000-220-010
5000-020-026
5000-020-027
5000-020-028
5000-020-029
(Reference only)

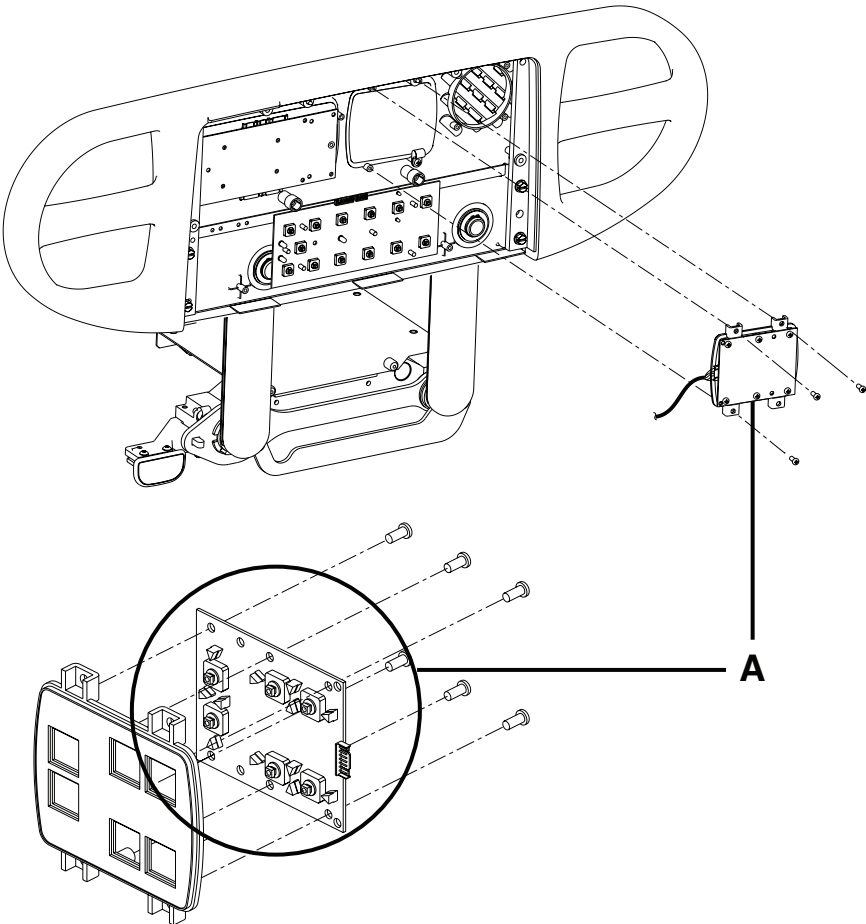

Item	Recycling/Material code	Important information	Qty
A	Lumbar Keypad PCB (5000-400-920 Right) and (5000-400-930 Left		1

Stryker France S.A.S.
ZAC – Avenue de Satolas Green
69330 Pusignan
France



Stryker Medical
3800 E. Centre Avenue
Portage, MI 49002
USA

stryker[®]