
IMPORTANT File in your maintenance records



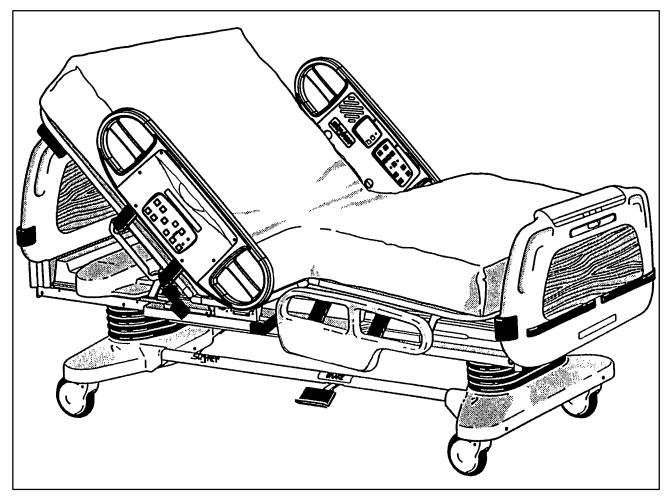

Secure 3000 Bed

MAINTENANCE MANUAL

For beds with serial numbers 970615095 through 971215116

For Parts or Technical Assistance 1–800–327–0770

Introduction
Specifications
Warning / Caution / Note Definition 4
Safety Tips And Guidelines
Limited Warranty
Obtaining Parts and Service
Return Authorization
Freight Damage Claims
Service Contract Coverage
Replacement Parts List
Preventative Maintenance
Cleaning
Maintenance Checklist
Set–Up Procedures
Bed Illustration
Brake Pedal Operation
Steer Pedal Operation
CPR Emergency Release Usage 14
Foley Bag Hooks Usage
Foot Prop Usage
Fracture Frame Usage
I.V. Poles
Night Light Usage
Patient Restraint Strap Locations
Chart Rack Usage
CPR Board Usage
Positioning Siderails
Siderail Control Panel Lights
Inside Siderail Function Guide
Outside Siderail Function Guide
Dynamic Mattress System Usage
Foot Board Control Panel Guide
Weigh System Operation Guide
Static Discharge Precautions
Troubleshooting
Troubleshooting Guide
Bed Component and Wiring Diagram
Bed Circuit Boards
Head Wall Output Configuration
Maintenance Procedures
Brake Pedal Replacement
Brake Sensor Replacement
Steer Wheel Cable Replacement and Adjustment
Caster Replacement
Lift Motor and Capacitor Removal and Replacement
Lift Motor Isolation Plate Replacement

Maintenance Procedures (Continued)	
Lift Housing Removal and Replacement	58, 59
Lift Potentiometer Replacement and Adjustment	60, 61
Lift Potentiometer "Burn-In" Procedure	61
Lift Motor Coupler Replacement	62
Power and Sensor Coil Cord Replacement	63, 64
Motion Interrupt Switch Replacement	
Knee Motor Removal and Replacement	
Head Motor Cam and Cam Guide Replacement	
Head Motor Limit Setting	
Head Motor Removal and Replacement	
Head Motor Drive Isolator and CPR Decoupler Removal and Replacement	
Head Motor Drive Screw and Ball Nut Replacement	
Weigh System Diagnostic Procedure	
Load Cell Replacement	
Power Board Replacement	
Litter CPU Board Replacement	
Main CPU Board "Burn–In" Procedure	
Siderail Cover Removal	
Molded Siderail Replacement	80
Head End Siderail Cable Replacement	
Siderail Assembly Removal	
Foot Board Hinge Removal	
Foot Board Module Replacement	
Foot Board Interface Plug Replacement	
Assembly Drawings and Parts Lists	
Base Assembly	
Lift Assembly (Common)	
Lift Assembly (Head and Foot End)	
Isolation Plate Assembly	
Potentiometer Assembly	
Brake Crank Assembly	
Brake Bar Assembly	
Manual Override Shaft Assembly	
Brake Shaft Assembly	
Bottom Base Cover Assembly	
Litter Assembly	
Electrical Cover Assembly	
Foot Cover Assembly	
Litter Assembly, Mechanical	
Gatch Motor Assembly	
Lift Header Assemblies	
Fowler Drive Assembly	
Fowler Limit Switch Assembly	
Litter Assembly, Electrical	
Power Plug Assembly	

Assembly Drawings and Parts Lists (Continued)	
Litter Assembly, Optional Scale and/or Bed Exit	
Outlet Box Assembly	
Litter Assembly, Optional 110V Outlet	
Litter Assembly, DMS Option	
Litter Assembly, Optional Night Light	
Litter Assembly, Optional HWI w/Communication and Ports	
Battery Enclosure Assembly	
Litter Assembly, Optional HWI Nurse Call	
Litter Assembly, Optional HWI Communication Capability	
Litter Assembly, Accessory Adapter Frame Option	
Optional 90° Power Cord Assembly	154
Head End Siderail Assembly	
Siderail Panel Assembly	
Head End Siderail Labels	
Foot End Siderail Assembly	
Siderail Support Assembly	
Siderail Release Lever Assembly	
Optional Pendant Assembly	
Foot Board Assembly	
Head Board Assembly	196
Optional CPR Board Assembly	197
Removable I.V. Pole Assembly	198
Removable I.V. Pole Mounting Assembly	199
Permanent I.V. Pole Mounting Assembly, Head End	200
Permanent I.V. Pole Mounting Assembly, Foot End	
Permanently Mounted I.V. Pole Assembly	
Litter Roller Option Assembly	
Bed Extender Assembly	
Optional Telephone Mounting Bracket Assembly	
Bed Assembly	

INTRODUCTION

This manual is designed to assist you with the maintenance of the Secure 3000 Bed. Read it thoroughly before beginning any maintenance on the equipment.

SPECIFICATIONS

Maximum Weight Capacity	500 pounds
Overall Bed Length/Width	93"/42–1/2"
Patient Sleep Surface	84"/35"
Minimum/Maximum Bed Height	16"/29–1/2"
Knee Gatch Angle	0° to 40°
Fowler Angle	0° to 60°
Trendelenberg/Reverse Trendelenberg	-11° to +11°
Weigh System Accuracy (optional equipment)	\pm 1% of total patient weight while the bed is level and up to a 4° or -4° angle \pm 3% of total patient weight from 5° to 11° angle or -5° to -11° angle
Electrical Requirements – all electrical requirements meet UL 544 specifications.	110 VAC, 60 Hz, 10.0 Amp.

WARNING / CAUTION / NOTE DEFINITION

The words WARNING, CAUTION and NOTE carry special meanings and should be carefully reviewed.

WARNING

The personal safety of the patient or user may be involved. Disregarding this information could result in injury to the patient or user.

CAUTION

These instructions point out special procedures or precautions that must be followed to avoid damaging the equipment.

NOTE

This provides special information to make important instructions clearer.

SAFETY TIPS AND GUIDELINES

Before operating the Secure Bed, it is important to read and understand all information in this manual. Carefully read and strictly follow the safety guidelines listed on this page.

It is important that all users have been trained and educated on the inherent hazards associated with the usage of electric beds.

WARNING

- The Secure 3000 Bed is not intended for use with patients less than two years of age.
- Powered bed mechanisms can cause serious injury. Operate bed only when all persons are clear of the mechanisms.
- To help reduce the number and severity of falls by patients, always leave the bed in the lowest position when the patient is unattended.
- Leave the siderails fully up and locked when the patient is unattended. When raising the siderails, listen
 for the "click" that indicates the siderail has locked in the up position. Pull firmly on the siderail to ensure
 it is locked into position.

Siderails are not intended to be a patient restraint device. It is the responsibility of the attending medical personnel to determine the degree of restraint necessary to ensure a patient will remain safely in bed.

- Always keep the caster brakes applied when a patient is on the bed (except during transport). Serious injury could result if the bed moves while a patient is getting in or out of bed. After the brake pedal is applied, push on the bed to ensure the brakes are locked. When moving the bed, toggle the steer pedal to put the bed in the steer mode. This locks the swivel motion of the right foot end caster and makes the bed easier to move.
- When large spills occur in the area of the circuit boards, 110 volt cables and motors, immediately unplug the bed power cord from the wall socket. Remove the patient from the bed and clean up the fluid. Have maintenance completely check the bed. Fluids can affect the operational capabilities of any electrical product. DO NOT put the bed back into service until it is completely dry and has been thoroughly tested for safe operation.
- Do not steam clean or hose off the bed. Do not immerse any part of the bed. The internal electric parts may be damaged by exposure to water. Hand wash all surfaces of the bed with warm water and mild detergent. Dry thoroughly. Quaternary Germicidal Disinfectants, used as directed, and/or Chlorine Bleach products, typically 5.25% Sodium Hypochlorite in dilutions ranging between 1 part bleach to 100 parts water, and 2 parts bleach to 100 parts water are **not** considered mild detergents. THESE PROD-UCTS ARE CORROSIVE IN NATURE AND MAY CAUSE DAMAGE TO YOUR BED IF USED IMPROP-ERLY. If these types of products are used to clean Stryker patient care equipment, measures must be taken to insure the beds are wiped with clean water and thoroughly dried following cleaning. Failure to properly rinse and dry the beds will leave a corrosive residue on the surface of the bed, possibly causing premature corrosion of critical components. Failure to follow the above directions when using these types of cleaners may void this product's warranty.

Clean Velcro **AFTER EACH USE**. Saturate Velcro with disinfectant and allow disinfectant to evaporate. (Appropriate disinfectant for nylon Velcro should be determined by the hospital.)

- Preventative maintenance should be performed at a minimum of biannually to ensure all bed features are functioning properly. Close attention should be given to safety features including, but not limited to: safety side latching mechanisms, frayed electrical cords and components, all electrical controls return to off or neutral position when released, caster braking systems, no controls or cabling entangled in bed mechanisms, leakage current 100 MA maximum, scale and bed exit systems calibrated properly.
- Always unplug bed during service or cleaning. When working under the bed with the bed in the high position, always place blocks under the litter frame and set the brakes to prevent injury in case the Bed Down switch is accidently pressed.

Limited Warranty:

Stryker Medical Division, a division of Stryker Corporation, warrants to the original purchaser that its products should be free from defects in material and workmanship for a period of either: one (1) year parts and labor **or** two (2) years for parts only, after date of delivery. Stryker's obligation under this warranty is expressly limited to supplying replacement parts for, or replacing, at its option, any product which is, in the sole discretion of Stryker, found to be defective. Stryker warrants to the original purchaser that the frame and welds on its beds will be free from structural defects for as long as the original purchaser owns the bed. If requested by Stryker, products or parts for which a warranty claim is made shall be returned prepaid to Stryker's factory. Any improper use or any alteration or repair by others in such manner as in Stryker's judgement affects the product materially and adversely shall void this warranty. No employee or representative of Stryker is authorized to change this warranty in any way.

This statement constitutes Stryker's entire warranty with respect to the aforesaid equipment. STRYKER MAKES NO OTHER WARRANTY OR REPRESENTATION, EITHER EXPRESSED OR IMPLIED, EXCEPT AS SET FORTH HEREIN. THERE IS NO WARRANTY OF MERCHANTABILITY AND THERE ARE NO WARRANTIES OF FITNESS FOR ANY PARTICULAR PURPOSE. IN NO EVENT SHALL STRYKER BE LIABLE HEREUNDER FOR INCIDENTAL OR CONSEQUENTIAL DAMAGES ARISING FROM OR IN ANY MANNER RELATED TO SALES OR USE OF ANY SUCH EQUIPMENT.

To Obtain Parts and Service:

Stryker products are supported by a nationwide network of dedicated Stryker Field Service Representatives. These representatives are factory trained, available locally, and carry a substantial spare parts inventory to minimize repair time. Simply call your local representative, or call Stryker Customer Service at (800) 327–0770.

Extended Warranty Coverage:

Stryker offers warranties to provide an extension of the above stated warranty, that are available upon the purchase of the Model 3000 Bed.

Covered under these warranties:

- All replacement parts, as set forth in the limited warranty statement above (excluding mattresses and consumable items)
 - NOTE: mattresses carry a separate warranty plan. Refer to mattress documentation.
- Labor and Travel for all scheduled and unscheduled calls (if labor option is chosen).

The following extended options are available at a nominal charge:

- 3 years parts no labor (1 year extension to standard 2 year parts warranty).
- 4 years parts no labor (2 year extension to standard 2 year parts warranty).
- 5 years parts no labor (3 year extension to standard 2 year parts warranty).
- 2 years parts and labor (1 year extension to standard 1 year parts and labor warranty).
- 3 years parts and labor (2 year extension to standard 1 year parts and labor warranty).
- 4 years parts and labor (3 year extension to standard 1 year parts and labor warranty).
- 5 years parts and labor (4 year extension to standard 1 year parts and labor warranty).

Return Authorization:

Merchandise cannot be returned without approval from the Stryker Customer Service Department. An authorization number will be provided which must be printed on the returned merchandise. Stryker reserves the right to charge shipping and restocking fees on returned items.

SPECIAL, MODIFIED, OR DISCONTINUED ITEMS NOT SUBJECT TO RETURN.

Damaged Merchandise:

ICC Regulations require that claims for damaged merchandise must be made with the carrier within fifteen (15) days of receipt of merchandise. DO NOT ACCEPT DAMAGED SHIPMENTS UNLESS SUCH DAMAGE IS NOTED ON THE DELIVERY RECEIPT AT THE TIME OF RECEIPT. Stryker Customer Service must be notified immediately. Stryker will aid the customer in filing a freight claim with the appropriate carrier for damages incurred. Claim will be limited in amount to the actual replacement cost. In the event that this information is not received by Stryker within the fifteen (15) day period following the delivery of the merchandise, or the damage was not noted on the delivery receipt at the time of receipt, the customer will be responsible for payment of the original invoice in full.

Claims for any short shipment must be made within thirty (30) days of invoice.

Service Contract Coverage (Optional):

Stryker has developed a comprehensive program of service contract options designed to keep your equipment operating at peak performance at the same time it eliminates unexpected costs. We recommend that these programs be activated *before* the expiration of the new product warranty to eliminate the potential of additional equipment upgrade charges. Stryker offers the following contract coverages at a nominal fee:

Extended (Parts and Labor)

- All replacement parts (excluding mattresses and consumable items)
- Labor and travel for all scheduled and unscheduled calls
- Biannual Preventive Maintenance Inspections and repairs
- JCAHO paperwork for preventive maintenance
- Priority Emergency Service

Standard (Labor Only):

- Labor and travel for all scheduled and unscheduled calls
- Biannual Preventive Maintenance Inspections and repairs
- JCAHO paperwork for preventive maintenance
- Priority Emergency Service

Basic (Parts Only):

- All replacement parts (excluding mattresses and consumable items)
- Priority Emergency Service

Please call your local representative, or call (800) 327–0770 for further information

ELECTRICAL COMPONENTS

DMS POWER SUPPLY/SERIAL IFC BOARD	3001–302–900
FOOT BOARD KEYBOARD (S/R LIGHTS, LOCKOUTS, ETC.)	3001–500–900
FOOT BOARD SCALE DISPLAY	3001–507–900
FOOT BOARD SCALE KEYBOARD	3001–507–910
FOOT BOARD BED EXIT KEYBOARD	3001–508–900
HEADWALL IFC W/O STRYKER PORT	3001–303–900
HEADWALL IFC W/STRYKER PORT	3001–314–900
LITTER CPU BOARD (NON-SCALE, NO BED EXIT)	3001–303–780
LITTER CPU BOARD (SCALE AND BED EXIT)	3001–307–783
POWER BOARD	3001–300–930
SIDERAIL BOARDS	
DMS BOARD	3001-402-900
INSIDE BOARD	3001–400–900
OUTSIDE BOARD	3001–400–910
SPEAKER W/CABLE	3000–403–831
OTHER COMPONENTS	
CAPACITOR, FOWLER	3000–300–453
CAPACITOR, GATCH	3000-300-401
CAPACITOR, LIFT	3000–200–243
CASTER, 6"	3000–200–30
CASTER, STEER, 6"	3000–200–16
CASTER, 8"	3000–200–36
CASTER, STEER, 8"	3000–200–37
COIL CORD, LIFT POWER	3001–200–824
COIL CORD, LIFT SENSOR	3001–200–815
COIL CORD, LITTER POWER	3001–300–844
COIL CORD, LITTER SIGNAL	3001–300–846
ISOLATION PLATE KIT, LIFT MOTOR	3000–200–723
LOAD CELL	3001–307–55
MOTOR COUPLER KIT, GATCH	3001–300–715
MOTOR COUPLER KIT, LIFT	3000–200–725
MOTOR, FOWLER W/CLUTCH	3001–300–703
MOTOR, GATCH	3001–300–435
MOTOR, LIFT (SAME FOR HEAD AND FOOT END)	3000–200–213
PADDED SIDERAIL COVER, HEAD, LEFT	3000–336–12
PADDED SIDERAIL COVER, HEAD, RIGHT	3000–336–11
PADDED SIDERAIL COVER, FOOT, LEFT	3000–336–14
PADDED SIDERAIL COVER, FOOT, RIGHT	3000–336–13
PADDED SIDERAIL COVERS, SET OF ALL FOUR	3000–336–701
POTENTIOMETER, FOOT END	3001–200–230
POTENTIOMETER, HEAD END	3000–200–240
POWER CORD	39–248
SINGLE TUBE OF GREASE	3000–200–700

UPGRADE KITS

UPGRADE KIT – ADD SCALE AND BED EXIT OPTIONS TO A STANDARD BED	3001–700–1
UPGRADE KIT – ADD SCALE OPTION TO A STANDARD BED	3001-700-2
UPGRADE KIT – ADD BED EXIT OPTION TO A STANDARD BED	3001-700-3
UPGRADE KIT – ADD SCALE OPTION TO BED WITH BED EXIT OPTION	3001-700-4
UPGRADE KIT – ADD BED EXIT OPTION TO BED WITH SCALE OPTION	3001-700-15

CLEANING

Hand wash all surfaces of the bed with warm water and mild detergent. Dry thoroughly.

CAUTION

Quaternary Germicidal Disinfectants, used as directed, and/or Chlorine Bleach products, typically 5.25% Sodium Hypochlorite in dilutions ranging between 1 part bleach to 100 parts water, and 2 parts bleach to 100 parts water are **not** considered mild detergents. THESE PRODUCTS ARE CORROSIVE IN NATURE AND MAY CAUSE DAMAGE TO YOUR BED IF USED IMPROPERLY. If these types of products are used to clean Stryker patient care equipment, measures must be taken to insure the beds are wiped with clean water and thoroughly dried following cleaning. Failure to properly rinse and dry the beds will leave a corrosive residue on the surface of the bed, possibly causing premature corrosion of critical components. Failure to follow the above directions when using these types of cleaners may void this product's warranty.

CAUTION

Do not steam clean or hose off the Secure 3000 Bed. Do not immerse any part of the bed. Some of the internal parts of the bed are electric and may be damaged by exposure to water.

Clean Velcro **AFTER EACH USE**. Saturate Velcro with disinfectant and allow disinfectant to evaporate. (Appropriate disinfectant for nylon Velcro should be determined by the hospital.)

In general, when used in those concentrations recommended by the manufacturer, either phenolic type or quaternary type disinfectants can be used with Staph–Chek fabrics. Iodophor type disinfectants are not recommended for use on Staph–Chek fabrics because staining may result. The following products have been tested by the Herculite Laboratory and have been found not to have a harmful effect on Staph–Chek fabrics WHEN USED IN ACCORDANCE WITH MANUFACTURERS RECOMMENDED DILUTION.*

TRADE NAME	DISINFECTANT TYPE	MANUFACTURER	*MANUFACTURER'S RECOMMENDED DILUTION
A33	Quaternary	Airwick (Professional Products Division)	2 ounces/gallon
A33 (dry)	Quaternary	Airwick (Professional Products Division)	1/2 ounce/gallon
Beaucoup	Phenolic	Huntington Laboratories	1 ounce/gallon
Blue Chip	Quaternary	S.C. Johnson	2 ounces/gallon
Elimstaph	Quaternary	Walter G. Legge	1 ounce/gallon
Franklin Phenomysan F2500	Phenolic	Purex Corporation	1 1/4 ounce/gallon
Franklin Sentinel	Quaternary	Purex Corporation	2 ounces/gallon
Galahad	Phenolic	Puritan Churchill Chemical Company	1 ounce/gallon
Hi–Tor	Quaternary	Huntington Laboratories	1/2 ounce/gallon
LPH	Phenolic	Vestal Laboratories	1/2 ounce/gallon
Matar	Phenolic	Huntington Laboratories	1/2 ounce/gallon
Omega	Quaternary	Airwick (Professional Products Division)	1/2 ounce/gallon
Quanto	Quaternary	Huntington Laboratories	1 ounce/gallon
Sanikleen	Quaternary	West Chemical Products	2 ounces/ gallon
Sanimaster II	Quaternary	Service Master	1 ounce/gallon
Vesphene	Phenolic	Vestal Laboratories	1 1/4 ounce/ gallon

REMOVAL OF IODINE COMPOUNDS

NOTE

This solution may be used to remove iodine stains from mattress cover and foam surfaces.

1. Use a solution of 1–2 tablespoons Sodium Thiosulfate in a pint of warm water to clean the stained area. Clean as soon as possible after staining occurs. If stains are not immediately removed, allow solution to soak or stand on the surface.

CHECKLIST (PERFORM A MINIMUM OF TWICE A YEAR)

- All fasteners secure (reference all assembly prints)
- All casters lock with brake pedal engaged
- "Brake Not Set" LED (on foot board) blinks when brakes are not engaged
- Locking steer caster engages and disengages properly
- _____ Siderails move, latch and stow properly
- _____ CPR release working properly
- Foot prop intact and working properly
- I.V. pole working properly
- Foley bag hooks intact
- Optional chart rack intact and working properly
- Optional CPR board not cracked or damaged and stores properly
- No cracks or splits in head and foot boards
- No rips or cracks in mattress cover
- All functions on head end siderails working properly (including LED's)
- All functions on footboard working properly (including LED's)
- Scale and Bed Exit system calibrated properly
- _____ Motion Interrupt switches working properly
- Optional night light working properly
- Power cord not frayed
- _____ No cables worn or pinched
- All electrical connections tight
- _____ All grounds secure to the frame
- _____ Ground impedance not more than 100 milliohms
- Current leakage not more than 100 microamps
- _____ Apply grease to litter grease points

Bed Serial No.	 	
Completed By:	Date:	

SET-UP PROCEDURES

It is important that the Secure 3000 Bed is working properly before it is put into service. The following list will help ensure that each part of the bed is tested.

• Plug the bed into a properly grounded, hospital grade wall receptacle and ensure the "Power" LED light at the foot end of the bed comes on.

WARNING

The 3000 is equipped with a hospital grade plug for protection against shock hazard. It must be plugged directly into a properly grounded three–prong receptacle. Grounding reliability can be achieved only when a hospital grade receptacle is used.

• Plug the optional interface cable into the 37 pin connector under the litter frame at the head end of the bed, and into the "Patient Station", "Head Wall", "Docker Station", or equivalent (whichever applies). Test the interface cable to verify it is functioning properly.

WARNING

Use only a Stryker supplied interface cable. Use of any other cable may cause the bed to function improperly which may result in patient or user injury.

- Ensure the siderails raise, lower and store smoothly and lock in the up and intermediate positions (page 18).
- Ensure that all four casters lock when the brake pedal is engaged (page 13).

NOTE

Ensure that the "Brake Not Set" LEDs located on the outside of the head end siderails and on the foot board control panel come on when the brakes are disengaged.

- Run through each function on the foot board control panel to ensure that each function is working properly (page 22–24).
- Run through each function on both head end siderails to ensure that each is working properly (page 19–21).
- Activate the motion stop system to ensure it is functioning properly: press and hold down the BED DOWN key. As the bed lowers, lift up on the motion interrupt pan (page 12) and ensure the downward motion stops. Release the pan and allow the downward motion to continue.

NOTE


The bed's <u>upward</u> motion or other functions are not disrupted by the motion stop system.

If any problems are found during bed set–up, contact Stryker Customer Service at 800–327–0770.

Damaged Merchandise

ICC Regulations require that claims for damaged merchandise must be made with the carrier within fifteen (15) days of receipt of merchandise. DO NOT ACCEPT DAMAGED SHIPMENTS UNLESS SUCH DAMAGE IS NOTED ON THE DELIVERY RECEIPT AT THE TIME OF RECEIPT. Stryker Customer Service must be notified immediately. Stryker will aid the customer in filing a freight claim with the appropriate carrier for damages incurred. Claim will be limited in amount to the actual replacement cost. In the event that this information is not received by Stryker within the fifteen (15) day period following the delivery of the merchandise, or the damage was not noted on the delivery receipt at the time of receipt, the customer will be responsible for payment of the original invoice in full.

Claims for any short shipment must be made within thirty (30) days of invoice.

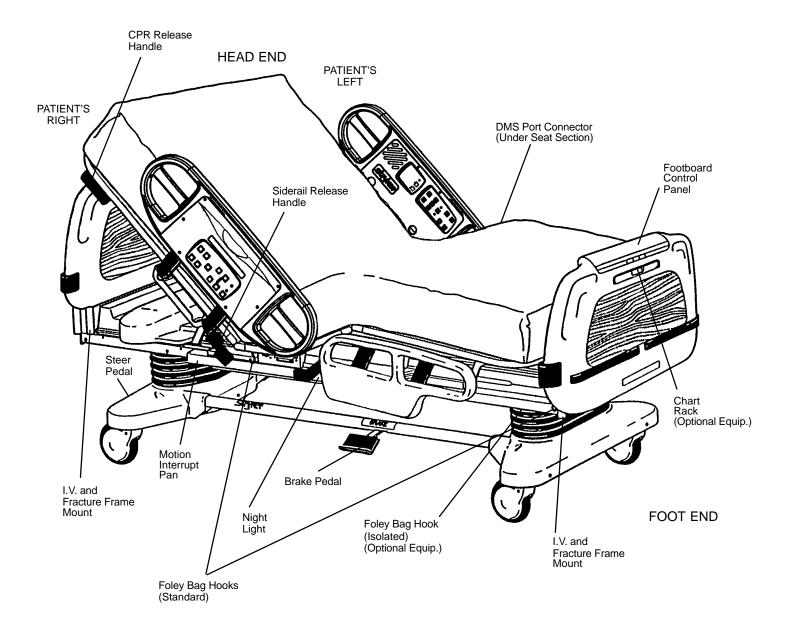

BRAKE PEDAL OPERATION

WARNING

Before putting a patient on the bed, be sure the brakes are fully engaged.

To activate the brakes, push down once on the pedal identified by the label at right (located at the midpoint of the bed on both sides). To disengage, push down once.



NOTE

There are LED lights on the outside of the head end siderails and on the foot end control panel that will blink when the brakes are not engaged only if the bed is plugged into a wall socket (see pages 20 & 22). The brakes will still operate properly when the bed is not plugged in.

STEER PEDAL OPERATION

The purpose of the steer caster is to help guide the bed along a straight line and to help with pivoting at corners when the bed is moved.

To activate the steer caster, move the pedal located at the head end of the bed to your left as shown on the label.



NOTE

For proper "tracking" of the steer caster, push the bed approximately 10 feet to allow the wheels to face the direction of travel before engaging the steer pedal. If this is not done, proper "tracking" will not occur and the bed will be difficult to steer.

CPR EMERGENCY RELEASE USAGE

When quick access to the patient is needed, and the Fowler is raised, squeeze one of the two red release handles (see illustration, page 12) and the Fowler can be guided down to a flat position.

NOTE

The handle can be released at any time to stop the Fowler from lowering.

FOLEY BAG HOOKS USAGE, STANDARD AND ISOLATED (Isolated Optional Equipment)

The standard Foley bag hooks are found at two locations on both sides of the bed, under the frame rail below the seat section and at the extreme foot end of the bed.

NOTE

The patient weight reading on the bed scale system will be affected by using the standard Foley bag hooks.

The optional isolated Foley bag hooks are located under the litter frame at the top of the foot end bellows.

CAUTION

The Foley bag hooks move when the Fowler is raised or lowered. Fowler motion must be locked out when using these hooks to avoid inadvertent movement of the hooks.

NOTE


The patient weight reading on the bed scale system <u>will not</u> be affected by usage of the isolated Foley bag hooks.

FOOT PROP USAGE

To prop the foot end of the Knee Gatch up, grasp the handle (A) at the end of the Knee Gatch and lift upward, allowing the latch arm to engage at the desired height. To release the prop, (B) lift up on the handle (A) and swing the foot prop (B) toward the head end of the bed to disengage the hinge and allow the foot end to lower.

WARNING


The intent of the foot prop is to elevate a patient's feet. To avoid injury while cleaning or servicing under the foot section, secure the foot section with string or bungee cords or hold it up out of the way.

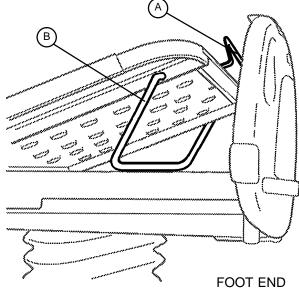

FRACTURE FRAME USAGE

A standard fracture frame can be mounted on the bed using the I.V. sockets located on all four corners of the bed. I.V. poles can be used in conjunction with a fracture frame if I. V. pole adaptor sockets are purchased.

I.V. POLES

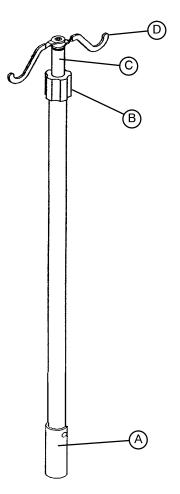

To use the Permanently Attached I.V. pole (optional equipment):

1. Lift and pivot the pole from the storage position and push down until it is locked into receptacle (A).

2. To raise the height of the pole, turn the lock actuator (B) counterclockwise and pull up on the telescoping portion (C) of the pole to raise it to the desired height.

3. Turn the lock actuator (B) clockwise to lock the telescoping portion in place.

4. Rotate the I.V. hangers (D) to desired position and hang I.V. bags.

CAUTION

The weight of the I.V. bags should not exceed 40 pounds.

To use the "Removable" I.V. pole (optional equipment):

1. Remove the pole from its storage position located at the foot end of the bed, under the foot board, or at the left side of the bed below the litter.


2. Install the pole at any of the six receptacles on the bed top (located on all four corners of the bed and at the midpoint of the bed, on both sides.)

3. To raise the height of the pole, turn knob (A) counterclockwise and pull up on the telescoping portion (B) of the pole and raise it to the desired height.

4. Turn knob (A) clockwise to tighten the telescoping portion in place.


CAUTION

The weight of the I.V. bags should not exceed 40 pounds.

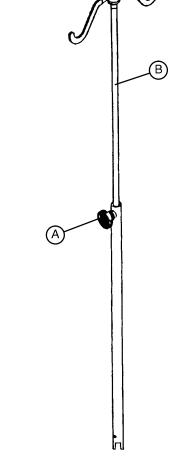

NIGHT LIGHT USAGE (Optional Equipment)

The bed may be equipped with an optional night light (A) that will illuminate the floor area around the bed. The light has three set– tings: LOW–OFF–HIGH.

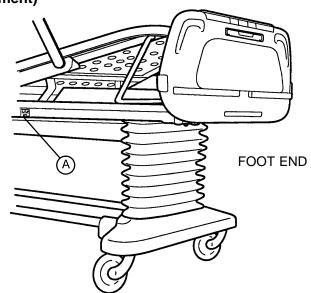

PATIENT RESTRAINT STRAP LOCATIONS

The bed is equipped with 12 separate locations for installing patient restraint straps. The "cutouts" in the bed top are located directly across from each other (on both sides of the bed).

WARNING

Improperly adjusted restraint straps can cause serious injury to a patient. The clinician must use her/his judgement to determine proper use of restraint straps and restraint strap locations.

Clean Velcro **AFTER EACH USE**. Saturate Velcro with disinfectant and allow disinfectant to evaporate. (Appropriate disinfectant for nylon Velcro should be determined by the hospital.)

NURSE CALL BACK-UP BATTERY

To prevent a low battery condition when the bed is not plugged in, position the cord out switch at the head end of the bed to the off position. The switch is identified by the label shown below. If the switch is not positioned as shown below and the bed power cord and pendant cord are unplugged, the life of the back-up battery will be significantly reduced.

If the power light (located on the foot board) is flashing, the Nurse Call battery needs to be replaced. The battery is located on the patient's left side at the head end of the bed. No tools are required to replace the battery. Unplug the bed power cord from the wall socket and replace the battery.


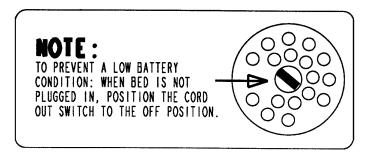


CHART RACK USAGE (Optional Equipment)

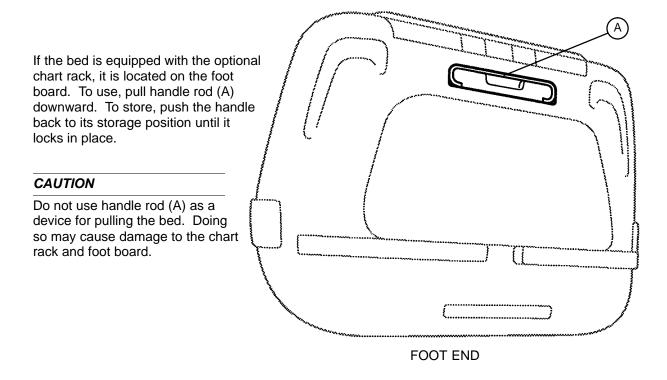

CPR BOARD USAGE (Optional Equipment)

If the bed is equipped with the optional CPR board, it is stored on the bed's head board. To remove, pull away from the head board and lift out of storage position. If the CPR board option was not purchased, the head board can also be removed and used as an emergency CPR board.

POSITIONING SIDERAILS

NOTE

The siderails can be locked at two heights (intermediate & full).


The siderails can be tucked away under the bed when not in use. To remove the rail from the tucked position, grasp the top of the rail and pull outward.

To engage the head end siderail, grasp the rail and swing it upward toward the head end of the bed until it rests in the "intermediate" position. To continue to full height, push in the blue release handle (A) and rotate the siderail until full height is reached.

To engage the foot end siderail, the same procedure is required as for the head end siderail, however, the siderail swings to the foot end of the bed.

WARNING

Be sure rail is locked securely into position. Siderails are not intended to keep patients from exiting the bed. They are designed to keep a patient from inadvertently rolling off the bed. Proper restraint methods should be utilized to ensure the patient remains in bed. The siderails are not intended to be used as a push device.

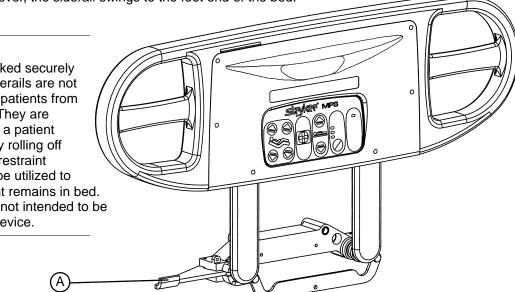

To disengage the rail, push in the blue release handle (A) and swing rail down to desired height. Tuck away siderails by pushing the rails under the bed. Rails must be in the full down position before they can be tucked.

SIDERAIL CONTROL PANEL LIGHTS

The bed is equipped with lights to illuminate the head end siderail control panel and the red nurse call switches. Both can be activated at the foot board control panel. Three settings are available for the control panel lights: low, medium and high intensity. When all lights are off, push the siderail control light button at the foot board once to turn on both the control lights and the nurse call light at the siderail. Push again to change from low to medium setting, and a third time to change to the high setting. The nurse call light intensity is not affected. Pushing the button a fourth time will turn off the siderail control panel lights and pushing it a fifth time will turn off the red nurse call light as well (see control panel guide page 22).

CAUTION

The intent of the red nurse call light on the siderails is to ensure the patient has immediate understanding of where to push to contact the nurse station. Turning this light off will compromise this ability, especially in a darkened room.

INSIDE SIDERAIL FUNCTION GUIDE













(Patient's Right Rail)

- 1. Push to raise Knee Gatch.
- 2. Push to lower Knee Gatch.
- 3. Push to raise Fowler.
- 4. Push to lower Fowler.

(Patient's Left Rail)

- 1. Push to raise Fowler.
- 2. Push to lower Fowler.
- 3. Push to raise Knee Gatch.
- 4. Push to lower Knee Gatch.
- 1. Push to activate Nurse Call.

NOTE


Yellow LED will light when button is pushed. Red LED will light with Nurse Station acknowledgment.

- ► This panel is optional equipment.
- 1. Push to turn TV or radio on and to select a channel.
- 2. Push to increase volume.
- 3. Push to decrease volume.
- ► This panel is optional equipment.

1. Push to increase firmness of mattress

2. Push to decrease firmness of mattress. (See page 21 for system instructions).


► This panel is optional equipment.

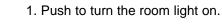

2. Push to turn the bed overhead light on.

► This panel is optional equipment.

OUTSIDE SIDERAIL FUNCTION GUIDE

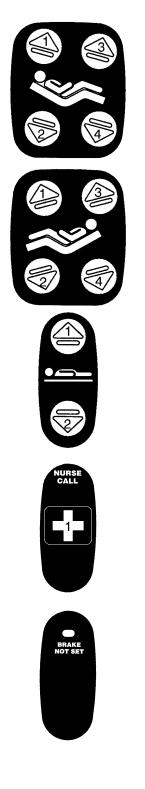

(Patient's Right Rail)

- 1. Push to raise Fowler.
- 2. Push to lower Fowler.
- 3. Push to raise Knee Gatch.
- 4. Push to lower Knee Gatch.
- ► This panel is optional equipment.

(Patient's Left Rail)

- 1. Push to raise Knee Gatch.
- 2. Push to lower Knee Gatch.
- 3. Push to raise Fowler.
- 4. Push to lower Fowler.
- ► This panel is optional equipment.
- 1. Push to raise bed height.

2. Push to lower bed height.

1. Push to activate Nurse Call.

► This panel is optional equipment.

LED will blink when the brakes are not set.

OUTSIDE SIDERAIL FUNCTION GUIDE (CONTINUED)



Push to activate auto or manual mode of the Dynamic Mattress System. LED will light to indicate selected mode and/or possible air leak.

► This panel is optional equipment.


DYNAMIC MATTRESS SYSTEM USAGE (OPTIONAL EQUIPMENT)

- The system can be set in either "AUTOMATIC" or "MANUAL" modes. The "AUTO/MAN" switch is used to activate both modes. When Automatic mode is selected, the "AUTOMATIC" LED on the siderail control panel will be on. When Manual mode is selected, the "MANUAL" LED on the siderail control panel will be on. The "AUTO/MAN" switches are located on the control panels on the outside of the head end siderails (see above).
- 2. When the Automatic mode is selected, the firmness of the mattress will adjust automatically as needed. When the Manual mode is selected, the firmness of the mattress can be adjusted by the patient or the hospital clinical staff. The "FIRM" and "SOFT" switches are located on the control panels on the inside of the head end siderails (see page 19).

NOTE

The mattress power cord must be plugged into the bed for the mattress to operate. See page 12 for "DMS Port" location.

FOOT BOARD CONTROL PANEL GUIDE

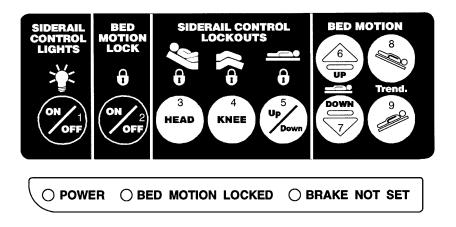

1. Push repeatedly for low, medium and high settings for siderail control lights. Pushing a fourth and fifth time will turn off the siderail control lights and the red nurse call light respectively (see page 18).

CAUTION

The intent of the red nurse call light on the siderails is to ensure the patient has immediate understanding of which button to push to contact the nurse station. Turning the red light off may compromise this ability, especially in a darkened room.

- 2. Push to lock out all bed motions. The MOTION lock icon and the "BED MOTION LOCKED" LED will light. Push again to unlock.
- 3. Push to lock out Back Rest controls at both siderails. The HEAD lock icon will light. Push again to unlock.
- 4. Push to lock out Knee Gatch controls at both siderails. The KNEE lock icon will light. Push again to unlock.
- 5. Push to lock out bed height movement at both siderails. The UP/DOWN lock icon will light. Push again to unlock.
- 6. Push to raise bed height.
- 7. Push to lower bed height.
- 8. Push to lower head end/raise foot end of bed (Trendelenberg position).
- 9. Push to lower foot end/raise head end of bed (Reverse Trendelenberg position).

FUNCTION LOCKOUT SYSTEM USAGE


1. To lock out the bed movement functions on the siderails and prevent the patient from changing the positioning of the bed, push the "HEAD", "KNEE" and/or "UP/DOWN" switches in the "Siderail Control Lockouts" module on the foot board control panel.

NOTE

The foot board controls for these motions are not affected by the lockout switches. The "padlock" symbol on the control panel will be lighted when that function is locked out.

2. To lock out the entire bed motion for all switches on the bed (siderails and foot board), push the "ON/OFF" switch in the "Bed Motion Lock" module on the foot board control panel.

FOOT BOARD CONTROL PANEL GUIDE (CONTINUED)

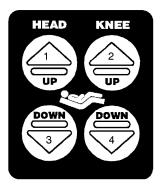

- 1. Push to raise Fowler.
- 2. Push to raise Knee Gatch.
- 3. Push to lower Fowler.
- 4. Push to lower Knee Gatch.
- ► This panel is optional equipment.

LED DISPLAY PANEL GUIDE

The LED Display Panel is located at the foot end of the bed, under the Control Panel.



"POWER" - will light when the bed is plugged into the wall receptacle.

"BED MOTION LOCKED" - will light when the Bed Motion Lock has been activated.

"BRAKE NOT SET" - will blink when the brakes have not been set.

"BED EXIT ON" - will light when the Bed Exit function has been activated (optional equipment).

CENTER OF GRAVITY BED EXIT (OPTIONAL EQUIPMENT)



1. Push to activate Bed Exit function.

2. Push to deactivate Bed Exit function.

NOTE


If the scale system is in use, it will switch to "off" when Bed Exit is armed. Bed Exit will be temporarily disarmed when the scale system is activated and will re–arm when the scale system shuts off. When the bed is equipped with scales, the scales must be properly zeroed for the Bed Exit System to function properly (see page 27 for scale system usage instructions). If bed is not equipped with a scale system, follow the procedure below.

- 1. Before putting a new patient on the bed: prepare bed for patient stay by adding linens and equipment to the bed. Press and <u>hold</u> "ARM" and "DISARM" keys together for 5 seconds "ARMED" light will begin to flash. Release "ARM" and "DISARM" keys and do not touch the bed until "ARMED" light stops flashing.
- 2. Once the new patient is on the bed: push "ARM" key and release ("ARMED" light will come on).
- 3. To deactivate Bed Exit, push "DISARM". The "ARMED" and "BED EXIT ON" LED's will turn off.

WARNING

The Bed Exit System is intended only to aid in the detection of a patient exiting the bed. It is NOT intended to replace patient monitoring protocol. It signals when a patient is about to exit. Adding or subtracting objects from the bed after arming the bed exit system may cause a reduction in the sensitivity of the bed exit system.

WEIGH SYSTEM CONTROL PANEL GUIDE

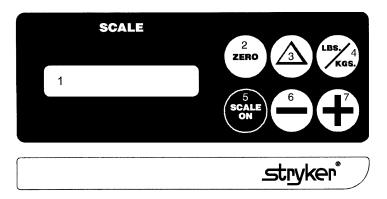

- 1. LCD displays patient weight.
- 2. Push to zero bed.
- 3. Push when changing equipment on the bed.
- 4. Push to change weight from pounds to kilograms or back.
- 5. Push to turn weight system on.
- 6. Push to decrease numerical value of displayed weight.
- 7. Push to increase numerical value of displayed weight.

NOTE

After approximately 30 seconds of idle time, the scale display will turn off and will show the Trendelenberg angle of the bed. Press "SCALE ON" to return to the weight display.

SYMBOL	ACTION	DISPLAY
	To prepare bed for new patient:	
SCALE	Press and hold "SCALE ON". Release the button <i>after</i> the display reads: "LET GO FOR SCALE"	"LET GO FOR SCALE" "WEIGHING" "XXX.X LB"
ZERO	Press and hold "ZERO" Release "ZERO"	"HOLD TO ZERO WT." "RELEASE TO ZERO" "DO NOT TOUCH BED" "0.0 LB"

WEIGH SYSTEM CONTROL PANEL GUIDE (CONTINUED)

SYMBOL	ACTION	DISPLAY
	To add or remove equipment during patient stay without affecting registered patient weight:	
	Press and release "SCALE ON"	"WEIGHING" "XXX.X LB"
(Δ)	Press Δ	"HOLD TO START" "RELEASE TO START"
	Release Δ	"DO NOT TOUCH BED" "ADD/REMOVE EQUIP."
	Add or remove equipment.	
	Press Δ	"RELEASE TO FIN."
	Release 🛆	"DO NOT TOUCH BED" "XXX.X LB"
	To convert the patient's weight:	
KGS.	To convert the patient's weight to kilograms, press and release "LBS./KG."	"WEIGHT NOW KGS" "XXX.X KG"
	Repeat the procedure to return to pounds.	
	To change the numerical value of displayed weight:	
	Press and hold to scroll to desired weight.	"HOLD TO DEC. WT." "XXX.X LB"
	Press h and hold to scroll to desired weight.	"HOLD TO INC. WT." "XXX.X LB"

OPERATING THE SCALE BEFORE PUTTING A NEW PATIENT IN BED

- Prepare bed for patient stay (linens, pillows, etc.).
- Press and hold "SCALE ON". Release the button *after* the display reads "LET GO FOR SCALE". (This will turn off the Trend. angle display and activate the scale). The scale monitor will read:

"LET GO FOR SCALE"

"WEIGHING"

"XXX.X LB"

• Press and hold "ZERO". The scale monitor will read:

"HOLD TO ZERO WT."

"RELEASE TO ZERO"

 Release "ZERO". The scale monitor will now read: "DO NOT TOUCH BED"

"0.0 LB"

The bed is now ready for the patient.

NOTE

Do not zero the bed while a patient is in bed. Inaccurate patient weight reading will result. If this should occur, remove the patient and zero the bed.

OPERATING THE SCALE IF PATIENT IS ALREADY IN BED

"HOLD TO START"

"RELEASE TO START"

• Release Δ . The scale monitor will read:

"DO NOT TOUCH BED"

"ADD/REMOVE EQUIP"

• Add or remove the equipment and press Δ . The scale monitor will read:

"RELEASE TO FIN."

• Release Δ . The scale monitor will read:

"DO NOT TOUCH BED"

"XXX.X LB"

The weight displayed will be that of the patient only.

CONVERTING THE PATIENT'S WEIGHT

• To convert the patient's weight from pounds to kilograms, press and release "SCALE ON" to activate the weigh system. After the scale monitor reads "XXX.X LB", press and release the "LBS/KGS" button. The scale monitor will read:

WEIGHT NOW KGS

"XXX.X KG"

• Repeat the procedure to return to pounds. The display will read: "WEIGHT NOW LBS"

"XXX.X LB"

CHANGING THE NUMERICAL VALUE OF DISPLAYED WEIGHT

• To decrease the numerical value of the displayed weight, press and hold "-". The scale monitor will read:

"HOLD TO DEC. WT."

"XXX.X LB"

- Hold "--" until desired value is achieved.
- To increase the numerical value of the displayed weight, press and hold "+". The scale monitor will read:

"HOLD TO INC. WT." "XXX.X LB"

• Hold "+" until desired value is achieved.

NOTE

The weigh system will shut off approximately one minute after a function has been used, if another function is not activated. Display light will shut off and display will read "SCALE OFF".

The weigh system will retain all patient weight information in its memory even when the scale monitor is off or when the bed is unplugged from the wall socket.

The electronic circuits in the 3000 are completely protected from static electricity damage only while the bed is assembled. It is extremely important that all service personnel always use adequate static protection when servicing the electronic systems of the 3000. *Whenever you are touching wires, you should be using static protection.*

Static Protection Equipment

The necessary equipment for proper static protection is:

- 1 static wrist strap; 3M part number 2214 or equivalent,
- 1 grounding plug; 3M part number 61038 or equivalent,
- 1 test lead with a banana plug on one end and an alligator clip on the other; Smith part number N132B699 or equivalent.

Stryker has available the following equipment for proper static protection:


- Complete static protection system part number 3000–000–753
- 1 grounding plug part number 3000–000–754
- 1 static wrist strap part number 3000–000–755
- 1 test lead part number 3000–000–756

CAUTION

All electronic service parts will be shipped in static shielding bags. Do not open the bags until you have completed steps 2 and 3 of the following procedure. Do not place unprotected circuit boards on the floor. All circuit boards to be returned to Stryker Medical should be shipped in the static shielding bags the new boards were shipped in.

Static Protection Procedure

- 1. Unplug the power cord from the wall receptacle.
- 2. Insert the grounding plug into a properly grounded hospital grade wall receptacle. Plug the banana plug of the test lead into the receptacle on the grounding plug. Connect the alligator clip on the other end of the test lead to a ground point on the bed.
- 3. Place the static control wrist strap on your wrist. Connect the alligator clip at the other end of the wrist strap cord to a ground point on the bed.

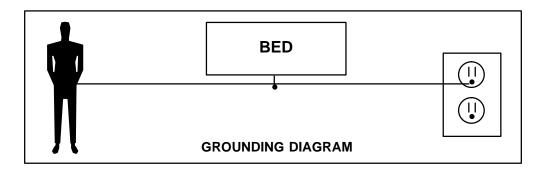

The Secure 3000 has only two main printed circuit boards in the litter (a CPU and a power PCB), two switch boards in each siderail, and one interface switch board at the foot board. Because of this design, isolating a "board failure" requires minimal troubleshooting time.

NOTE

Additional option boards may include a head wall interface board in the litter for Nurse Call, a Bed Exit switch board, a scale switch board and LCD at the foot board control panel for Bed Exit and Scale options, a DMS power supply and interface board in the litter, and a DMS switch board in each siderail for the DMS mattress option.

A general reason for troubleshooting a bed is to isolate a problem to the appropriate PCB. To do this as quickly as possible, it is important to understand where the boards and cabling are located and what the responsibility and voltage requirements are for each connector on the boards. The following troubleshooting section is very valuable if used properly. Pages 34–39 display an overhead view of the litter to locate and identify all electrical components and wiring and the replacement part numbers for each component. Pages 42–47 show the litter circuit boards and identification of each connector, the cabling location as it is identified on the board and the corresponding cable, the proper voltage for each pin, where to attach the voltmeter's positive and negative leads and what the voltage represents (description). The description column will guide you to the appropriate connector for voltage verification.

MAIN LITTER BOARDS

CPU BOARD P/N 3001–303–770 (No Scale or Bed Exit), P/N 3001–307–772 (Scale & Bed Exit), P/N 3001–308–770 (Bed Exit Only), P/N 3001–307–770 (Scale Only)

The CPU/control board is the "mother board" of the bed. It contains no primary voltages. It contains secondary voltages for the foot board, siderails, Nurse Call interface, downward motion interrupt system, weigh system load cells, and switch board signalling for the relays on the power board for electrical motion. It also contains all the bed's logic capabilities and limit information.

POWER BOARD P/N 3001-300-930

This board contains primary voltages. It receives secondary voltage signals from the CPU to activate its relays for bed motion. It also receives the main power from the power cord and provides the power for the Fowler, Knee Gatch, Bed Lift motors, 110V outlet, Night Light, and DMS power board. It has a transformer and voltage regulators and also contains the bed's main power fuses.

SIDERAILS

Each siderail contains an outside and an inside switch board. Power to the siderails is provided by the CPU board. Determine if the problem exists only in one siderail or in both siderails.

A.) If the problem is isolated to one siderail, switch the siderail connectors on the CPU. If the problem is transferred to the other siderail, and the "bad" rail is now functional, the CPU needs to be replaced. If the problem is not passed to the other rail and the "bad" rail is still non–functional, verify the cable connections in the siderail are intact. If the connections are intact, replace the "bad" siderail's switch board.

B.) If the problem exists in both siderails, verify the voltages at the CPU for the siderails. If the proper voltages are not present, replace the CPU. If the proper voltages are present, the siderail boards are probably O.K. and troubleshooting the individual component (head motor, knee motor, etc.) for the function is required.

FOOT BOARD

A full option foot board contains the main foot board PCB, a bed exit board, a scale switch board, and a scale LCD. Power to the foot board is provided by the CPU. Determine if the problem exists only at the foot board or also at the siderails.

A.) If the problem exists only at the foot board, use a foot board from another bed and verify that the problem persists. If the problem is gone when the foot board is changed, replace the non–functional module on the original foot board.

NOTE

The main foot board PCB (part number 3001–500–900) has interface responsibilities. All switch boards on the foot board are connected to this board. Verify the connections are intact. If they are, the problem could be either the main foot board computer board or the switch board.

If the problem does not go away with another foot board, verify the voltages to the foot board on the CPU. If the proper voltages are not present, replace the CPU. If the voltages are present, check the cable from the CPU to the foot board receptacle. Verify the foot board receptacle pins on the litter are intact.

B.) If the problem exists not only at the foot board but also at the siderails, troubleshooting the individual component (head motor, knee motor, etc.) for the function is required.

BED LIFT

The height of the bed is adjusted by a separate lift motor on each end of the bed in the lift unit. The motor is coupled with a series of gears that run the lift screws to move the litter up or down. When the bed lift or Trendelenburg button on either the foot board or siderail is pushed, the signal passes through the CPU and into the power board, activating a relay on the power board to energize the lift motor(s) from the power board. The motor limits are held by a potentiometer (pot) on each lift unit. As the lift screws turn, the pot is turned. As the voltage changes across the pot, the CPU recognizes when to shut off the motor signal at both the high and the low end.

A.) If either lift motor electrically stops prematurely, or continues to rum past its limit, try "burning–in" the lift pots (see page 78). If the problem persists, check voltage at the CPU with the bed in the full up position. If the voltage is too high, the pot has moved out of calibration and requires resetting. if resetting the pot does not solve the problem, the lift pot probably needs to be replaced (see pages 60 & 61).

B.) If one of the lift motors does not run at all, listen for a relay "click" on the power board when the button is pressed. If the relay is activating, try "burning–in" the lift pots (see page 78). If the "burn–in" procedure doesn't help, either the power board is not providing proper voltage to the motor, the cable is not transferring power to the motor, the motor capacitor has failed, or the motor itself has failed. Check all of these factors to isolate the problem.

If a relay is not being activated when the button is pushed, check the secondary voltage signals from the CPU at the power board. If proper voltage is present at the power board, replace the power board (see page 77). If proper voltage is not present at the power board, replace the CPU board (see page 78).

FOWLER AND KNEE LIFT

The heights of the Fowler and Knee are adjusted by separate motors located directly under the Fowler and the Knee. The motors are coupled to a worm gear that moves the Fowler or Knee up or down. When the Fowler or Knee button on either the foot board or siderail is pressed, the signal passes through the CPU and into the power board, activating a relay on the power board, and energizing the Fowler or Knee motor from the power board. The motor limits are held using a cam and micro switches. As the Fowler or Knee moves, the cam is pulled or pushed until a micro switch on both the high and low end of travel is activated.

A.) If the motor electrically stops prematurely or continues to run after its limit is reached, an adjustment of the cam is probably required (see page 68). If the switch still does not shut off the motor after the adjustment is made, either the switch is defective, the cabling from the switch is defective or not connected properly, or the switch is not being recognized by the CPU board. Check all of these factors to isolate the problem.

B.) If one of the motors is not running at all, listen for a relay "click" on the power board when the appropriate button is pressed. If the relay is activating, either the power board is not providing proper voltage to the motor, the cable is not transferring power to the motor, the motor capacitor has failed, or the motor has failed. Check all of these factors to isolate the problem.

If a relay is not being activated when the button is pressed, check the secondary voltage signals from the CPU at the power board. If the proper voltage is present at the power board, replace the power board (see page 77). If the proper voltage is not present at the power board, replace the CPU board (see page 78).

SCALE/BED EXIT SYSTEM

Power and logic for the scale and bed exit system comes from the CPU. Since the most common service requests on the scale system are identified as "user issues", the first step in troubleshooting a scale problem is understanding how the system is used (see pages – for scale system usage instructions). Rule out user issues by "zeroing" the bed and testing the scale system with a known weight. Verify the change equipment function is working. If the system checks O.K., it can be assumed the system was not being used properly.

If the system does not weigh within tolerance, re–calibrate the weigh system (see page 75). If, after calibration, the system still does not weigh accurately, look for a possible defective load cell (see page 75). If a defective load cell is detected, replace the load cell (see page 76 & 77). If the load cells are not defective, replace the CPU board (see page 78).

DMS – DYNAMIC MATTRESS SYSTEM

The integrated DMS mattress system is plugged into the bed and gets its power from the DMS power supply board. The DMS power supply board gets its power from the bed's power board. All of the logic for the mattress is contained in the control box within the mattress. When the mattress is plugged into the bed, the DMS LED will illuminate on the outside of each siderail. Rule out user issues by verifying the mattress is being used properly (see page 21 for DMS usage instructions).

A.) The controls for the DMS are integrated in each siderail. If a problem exists with the controls, isolate the problem to the siderails (see page 31).

B.) If the DMS indicator LED for manual or automatic does not illuminate and the mattress pump does not work when the mattress is plugged into the bed, verify the mattress is receiving power by checking incoming and outgoing voltage at the DMS power supply board. If voltage is present, the controller unit in the mattress will probably need replacing. If voltage is not present going into the DMS power supply board, either the cable connections at the board, the cabling coming from the bed's power board, or the bed power cord itself are defective. Check all these factors to isolate the problem. If voltage is present going into the DMS power supply board but not coming out, replace the DMS power supply board.

C.) If the Air Loss LED comes on, unzip the mattress cover and check for holes in the bladder or air lines or for loose air lines. If everything is intact, probably the DMS control unit in the mattress needs to be replaced.

NURSE CALL

A full side communication package includes switches in both siderails for interfacing with Nurse Call, TV/Volume, and Room/Read Lights. When the switches are activated, the signalling is passed through the CPU and into the head wall interface board which contains the 37 pin connector for the interface cable between the bed and the wall system. The signal is a momentary switch closure and does not contain voltage. Stryker is responsible for the successful interface between the bed and the existing system. Stryker is not responsible for the nurse call system installation or modifications to the current system. Within reason, whatever your current system or soon to be installed system does, Stryker will manufacture an interface cable that will accommodate a direct hookup to the system. (Full option will interface with Nurse Call, TV, Room and/or Read Lights).


NOTE


If the nurse call system is being modified to accommodate Room/Read Light controls, a step down voltage supply should be no greater than 30 volts for the bed to successfully interface with the lights.

If pin out information is required for the bed's 37 pin connector, see page 48.

The head wall interface board has changeable jumpers at locations J2, J3, J4, J8, HDR 12, and HDR 13. J2 is in line with Custom Pendant Signalling J3 is in line with Nurse Call Signalling J4 is in line with Priority Signalling J8 is in line with Pot Wiper for Speakers HDR 12 is in line with Priority Signalling HDR 13 is a storage location for jumpers

The purpose of the jumpers is to allow Stryker the opportunity to manufacture cables with fewer internal jumpers. The jumpers are on the board to create either a normally open or normally closed environment depending on the system's requirements. J2–J8 have three pins with a two pin jumper on two of the pins. A pin on the right or left will always be open. HDR 12 and HDR 13 have four pins each and will either have all pins jumped or none, whichever the system requires. Stryker will build the beds with the appropriate jumper locations utilized for the hospital's system. If the systems are changed in the future, it may be necessary to change jumper locations. Call Stryker's Technical Services for assistance. Stryker maintains records showing which systems are in use at individual accounts and how the cables should be configured. Please order replacement cables from Stryker.

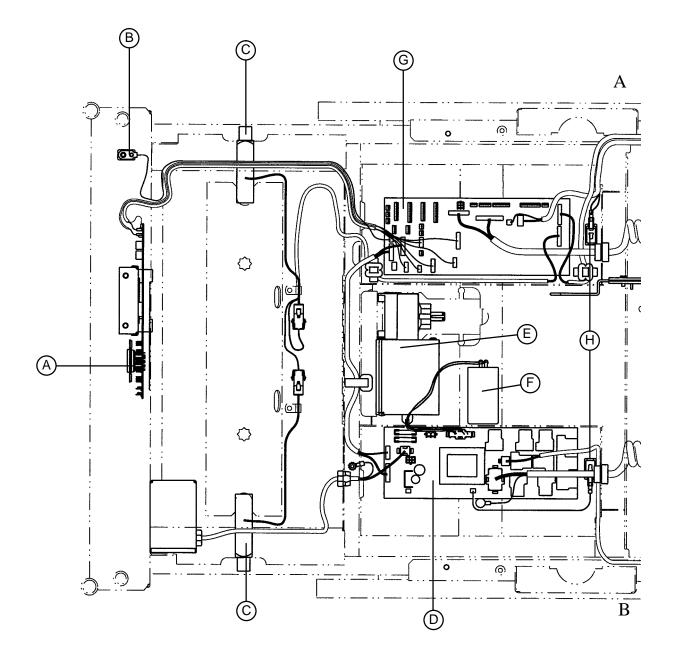


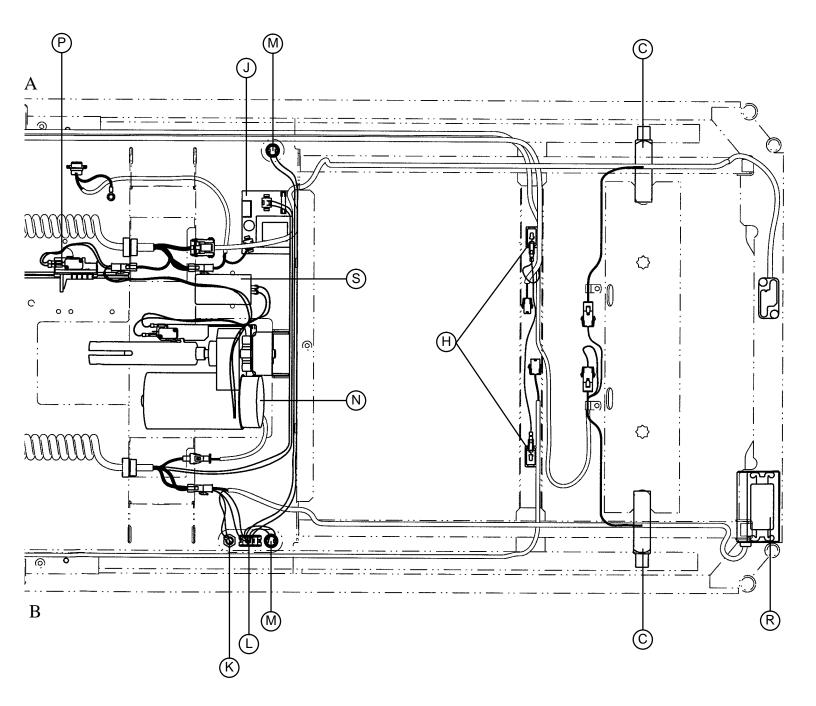


Bed Component Diagram

ltem	Part No.	Part Name
А	3001-303-900	Head Wall IFC Board
	3001–314–900	Head Wall IFC Board w/Stryker Port
В	3000–303–871	9V Battery (Nurse Call Backup)
С	3001–307–55	Load Cell
D	3001–300–930	Power Board
E	3001–300–703	Fowler Motor
F	3000–300–453	Fowler Capacitor
G	3001–307–783	CPU Board Kit (Scale and Bed Exit)
	3001–303–780	CPU Board Kit (Non–Scale, No Bed Exit)
Н	3000–300–58	Motion Interrupt Switches
J	3001-302-900	DMS Power Supply
K	59–770	110V Outlet Fuse (3 Amp)
L	3000–310–828	Night Light Switch
Μ	59–798	Night Light Bulb
Ν	3001–300–410	Gatch Motor
Р	3000–300–41	Fowler Limit Switch
R	59–733	110V Outlet
S	3000–300–401	Gatch Capacitor


Notes

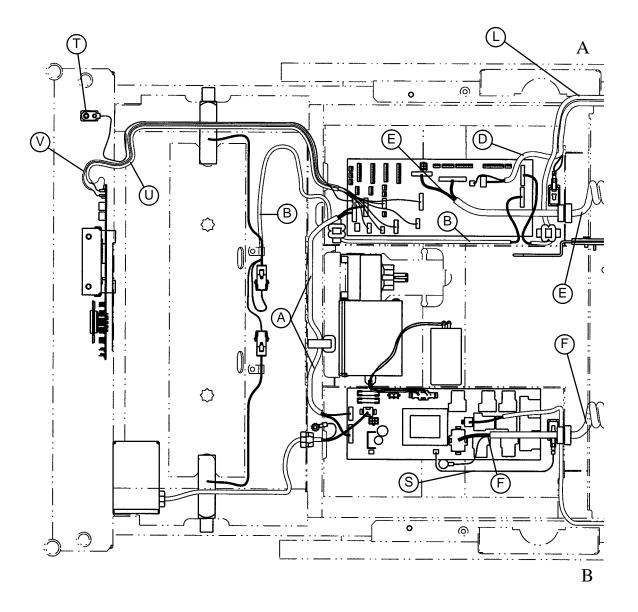

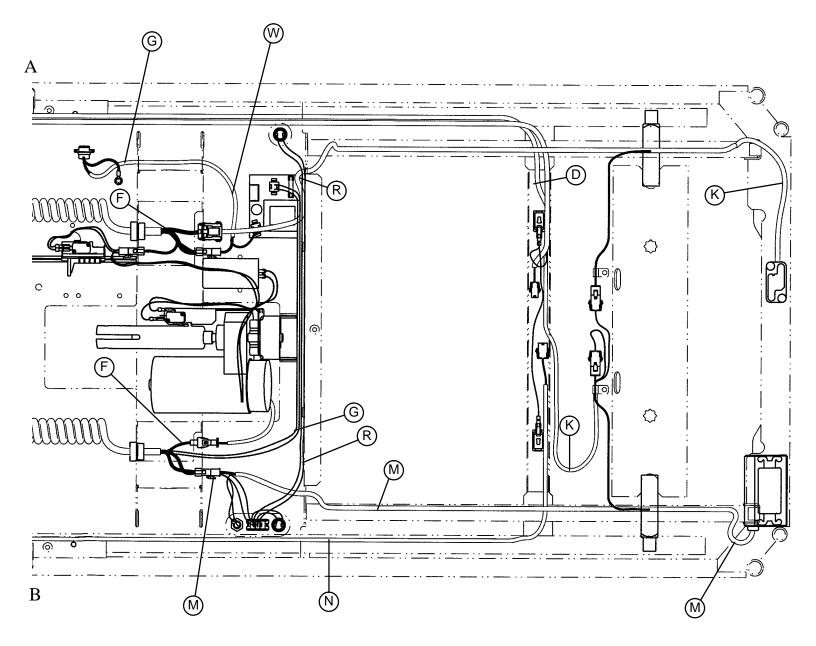


ltem	Part No.	Part Name
А	3001-300-804	Power PCB/CPU Control Cable
В	3001–307–812	Head End Load Cell Cable
D	3001–300–805	Litter Pot Cable (Foot End Bed Lift Pot & Head & Foot End Motion Interrupt Switches)
E	3001–300–846	Litter Signal Coil Cord (Fowler & Knee Cam Limits, Foot Board Controls)
F	3001–300–844	Litter Power Coil Cord (Knee Motor, Night Light, 110V Outlet, DMS)
G	3001–302–822	DMS Port Cable
Н	3001–300–809	Limit Switch Cable (Fowler & Knee Motor Limits)
K	3001–300–833	Foot Board Receptacle Cable
L	3001–307–813	Foot End Load Cell Cable
Μ	3001–320–801	110V Outlet Cable
Ν	3001-300-807	Foot End Bed Lift Cable (Foot End Bed Lift Motor)
R	3001–310–801	Night Light Cable
S	3001–300–810	Motion Interrupt Switch Cable
Т	3001–303–801	9V Battery Cable
U	3001–314–801	Cable Pendant Port Option
V	3001–303–800	HWI to CPU Cable
W	3001–302–821	DMS Signal Cable

Bed Wiring Diagram

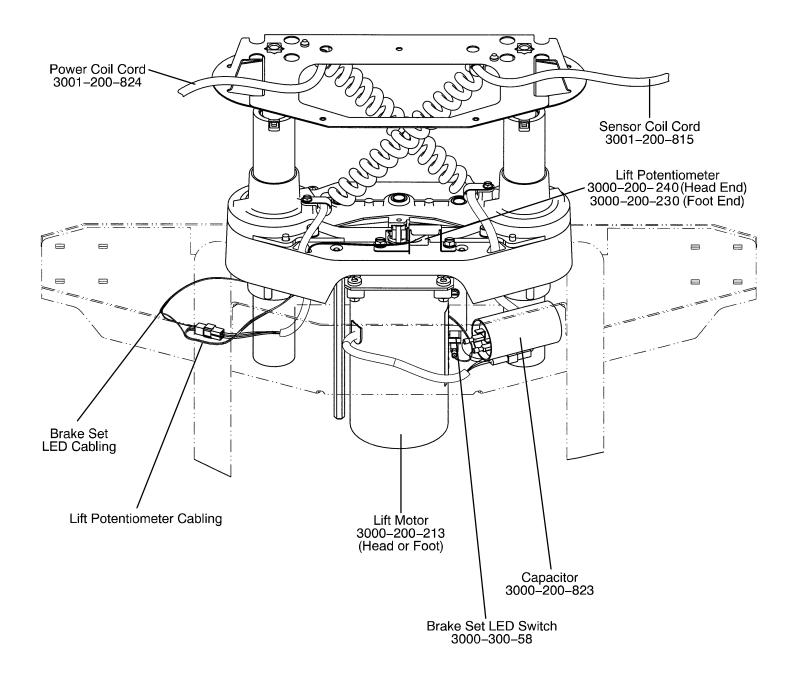

CPU BOARD PCB P/N 3001–303–770 – NO SCALE OR BED EXIT OPTIONS P/N 3001–307–770 – SCALE OPTION ONLY P/N 3001–307–772 – ALL OPTIONS P/N 3001–308–770 – BED EXIT OPTION ONLY

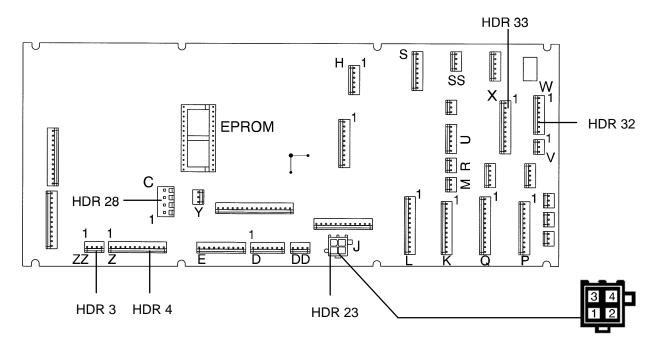


CONNECTOR LOCATION	CABLE LOCATION	VOLTAGE	POSITIVE LEAD	NEGATIVE LEAD	DESCRIPTION
HDR 12	A	+5 VDC	Pin 12	HDR 33 Pin 1	Foot End Load Cells
HDR 12	A	-5 VDC	Pin 11	HDR 33 Pin 1	Foot End Load Cells
HDR 12	A	+5 VDC	Pin 6	HDR 33 Pin 1	Foot End Load Cells
HDR 12	A	–5 VDC	Pin 5	HDR 33 Pin 1	Foot End Load Cells
HDR 21	F	+5 VDC	Pin 12	HDR 22 Pin 6	Foot Board
HDR 22	FF	+5 VDC	Pin 12	Pin 6	Foot Board
HDR 22	FF	+5 VDC	Pin 10	Pin 6	Foot Board
HDR 22	FF	+5 VDC	Pin 9	Pin 6	Foot Board
HDR 22	FF	+5 VDC	Pin 8	Pin 6	Foot Board
HDR 22	FF	16–18 VDC	Pin 7	Pin 6	Foot Board

CPU BOARD PCB P/N 3001–303–770 – NO SCALE OR BED EXIT OPTIONS P/N 3001–307–770 – SCALE OPTION ONLY P/N 3001–307–772 – ALL OPTIONS P/N 3001–308–770 – BED EXIT OPTION ONLY

CONNECTOR LOCATION	CABLE LOCATION	VOLTAGE	POSITIVE LEAD	NEGATIVE LEAD	DESCRIPTION
HDR 24	Y	16–18 VDC w/o Switch	Pin 1	HDR 33 Pin 1	Motion Interrupt
HDR 24	Y	0 VDC w/Switch	Pin 1	HDR 33 Pin 1	Motion Interrupt
HDR 32	W	0 VDC w/Switch 16–18 VDC wo/Switch	Pin 7	Pin 1 HDR 33	Enables Foot/Down
HDR 32	W	0 VDC w/Switch 16–18 VDC w/o Switch	Pin 8	Pin 1 HDR 33	Enables Foot/Head Down
HDR 32	W	5 VDC w/switch 0 VDC w/o switch	Pin 4	Pin 1 HDR 33	Power Board (Foot End Bed Up)
HDR 32	W	5 VDC w/switch 0 VDC w/o switch	Pin 3	Pin 1 HDR 33	Power Board (Foot End Bed Down)
HDR 32	W	5 VDC w/switch 0 VDC w/o switch	Pin 2	Pin 1 HDR 33	Power Board (Fowler Up)
HDR 32	W	5 VDC w/switch 0 VDC w/o switch	Pin 1	Pin 1 HDR 33	Power Board (Fowler Down)
HDR 33	Х	5 VDC w/switch 0 VDC w/o switch	Pin 11	Pin 1 HDR 33	Power Board (Gatch Up)
HDR 33	Х	5 VDC w/switch 0 VDC w/o switch	Pin 10	Pin 1 HDR 33	Power Board (Gatch Down)
HDR 33	Х	5 VDC w/switch 0 VDC w/o switch	Pin 9	Pin 1 HDR 33	Power Board (Safety Relay – all Motions)
HDR 33	Х	0 VDC wo/Switch 5 VDC w/Switch	Pin 8	Pin 1 HDR 33	Power Board (Motion Interrupt)
HDR 33	X	–5 VDC	Pin 7	Pin 1 HDR 33	Power Board
HDR 33	Х	16–18 VDC	Pin 6	Pin 1 HDR 33	Power Board
HDR 33	Х	16–18 VDC	Pin 5	Pin 1 HDR 33	Power Board
HDR 33	Х	5 VDC	Pin 4	Pin 1 HDR 33	Power Board

POWER BOARD PCB - P/N 3001-300-930

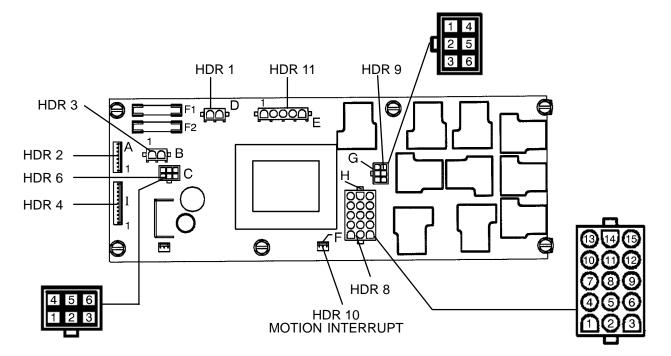

F1 – Fuse 6 Amp, 250 VAC, SLO–BLO – Stryker Part Number 59–149


F2 – Fuse .3 Amp, 250 VAC, SLO-BLO – Stryker Part Number 59-730

CONNECTOR LOCATION	CABLE LOCATION	VOLTAGE	POSITIVE LEAD	NEGATIVE LEAD	DESCRIPTION
HDR 2	A	0 VDC w/o Switch +5 VDC w/Switch	Pin 6	Pin1 HDR 4	Head End Bed Up
HDR 2	A	0 VDC w/o Switch +5 VDC w/Switch	Pin 5	Pin1 HDR 4	Head End Bed Down
HDR 2	A	0 VDC w/o Switch +5 VDC w/Switch	Pin 4	Pin1 HDR 4	Foot End Bed Up
HDR 2	A	0 VDC w/o Switch +5 VDC w/Switch	Pin 3	Pin 1 HDR 4	Foot End Bed Down
HDR 2	A	0 VDC w/o Switch +5 VDC w/Switch	Pin 2	Pin 1 HDR 4	Fowler Up
HDR 2	A	0 VDC w/o Switch +5 VDC w/Switch	Pin 1	Pin 1 HDR 4	Fowler Down
HDR 4	Ι	0 VDC w/o Switch +5 VDC w/Switch	Pin 11	Pin 1 HDR 4	Gatch Up
HDR 4	Ι	0 VDC w/o Switch +5 VDC w/Switch	Pin 10	Pin 1 HDR 4	Gatch Down
HDR 4	Ι	0 VDC w/o Switch +5 VDC w/Switch	Pin 9	Pin 1 HDR 4	Safety Relay
HDR 4	Ι	0 VDC w/o Switch +5 VDC w/Switch	Pin 8	Pin 1 HDR 4	Motion Interrupt

POWER BOARD – P/N 3001–300–930 (Continued)

CONNECTOR LOCATION	CABLE LOCATION	VOLTAGE	POSITIVE LEAD	NEGATIVE LEAD	DESCRIPTION
HDR 4	Ι	–5 VDC	Pin 7	Pin 1 HDR 4	CPU
HDR 4	Ι	+16 - +18 VDC	Pin 6	Pin 1 HDR 4	CPU
HDR 4	Ι	+16 – +18 VDC	Pin 5	Pin 1 HDR 4	CPU
HDR 4	Ι	5 VDC	Pin 4	Pin 1 HDR 4	CPU
HDR 6	С	0 VDC w/o Switch +5 VDC w/Switch	Pin 6	Neutral Pin 1	Head End Bed Motor Up
HDR 6	С	0 VDC w/o Switch +5 VDC w/Switch	Pin 3	Neutral Pin 1	Head End Bed Motor Down
HDR 9	G	0 VDC w/o Switch +5 VDC w/Switch	Pin 6	Neutral Pin 1	Foot End Bed Motor Up
HDR 9	G	0 VDC w/o Switch +5 VDC w/Switch	Pin 3	Neutral Pin 1	Foot End Bed Motor Down
HDR 11	E	0 VAC w/o Switch 110 VAC w/Switch	Pin 1	Neutral Pin 3	Fowler Motor Up
HDR 11	E	0 VAC w/o Switch 110 VAC w/Switch	Pin 2	Neutral Pin 3	Fowler Motor Down
HDR 8	Н	0 VAC w/o Switch 110 VAC w/Switch	Pin 10	Neutral Pin 15	Gatch Motor Up
HDR 8	Н	0 VAC w/o Switch 110 VAC w/Switch	Pin 9	Neutral Pin 15	Gatch Motor Down
HDR 3	В	110 VAC	Pin 2	Pin 1	Power Cord
Fuse 1		10A 3AG slo blo			Main Power Fuse
Fuse 2		3/10A 3AG slo blo			Main Power Fuse

DMS POWER SUPPLY PCB – P/N 3001–302–910

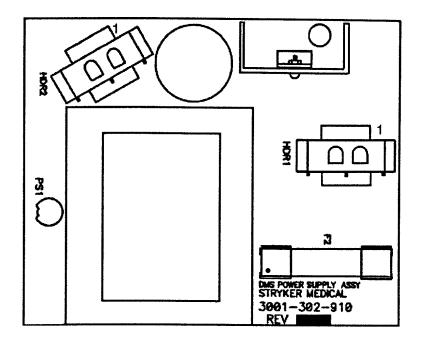


PS1 – Resettable Poly Switch

F2 – Fuse – 3/10 Amp, 3 AG, 250 VAC, SLO–BLO – Stryker Part Number 59–73


CONNECTOR LOCATION	CABLE LOCATION	VOLTAGE	POSITIVE LEAD	NEGATIVE LEAD	DESCRIPTION
HDR 1	See above	120 VAC	Pin 1	Pin 2	DMS 120 VAC Supply Voltage Neutral
HDR 2	See above	+12 VDC	Pin 2	Pin 1	DMS +12 VDC Power Supply Voltage

HEADWALL INTERFACE PCB – P/N 3001–303–900 HEADWALL INTERFACE PCB W/STRYKER PORT – P/N 3001–314–900

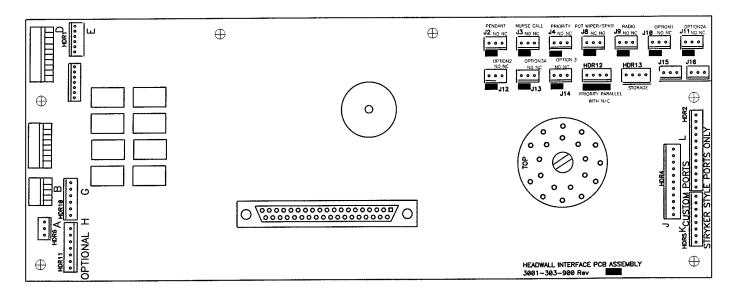

CONNECTOR LOCATION	CABLE LOCATION	VOLTAGE	POSITIVE LEAD	NEGATIVE LEAD	DESCRIPTION
HDR 7	F	0 VDC w/o Beeper On +5 VDC w/Beeper Energized	Pin 5	HDR 8–6	Beeper
HDR 8	С	GND	N/A	Pin 6	DC Ground
HDR 8	С	+16 V – +18 VDC	Pin 7	Pin 6	Bulk DC
HDR 8	С	+5 V	Pin 8	Pin 6	+5 VDC from Power Source
J5	DB 37 Pin Connector No Cable Designator	Nurse Call +	N/A A continuity check be- tween J5–25 & J5–26 for normally closed nurse call system will be positive when the nurse call but- ton is pushed	N/A	Nurse Call System +
J5	DB 37 Pin Connector No Cable Designator	Nurse Call No/NC Return J3 Header allows for a select nor- mally open or nor- mally closed NC system	N/A For normally open systems it will be posi- tive when the nurse call but- ton is not pushed	N/A	Nurse Call System + Return

37–PIN CONNECTOR

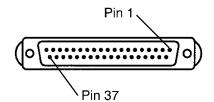

Pin 1	Option 2 Common
Pin 2	Read Light
Pin 3	Room Light
Pin 4	Speaker High
Pin 5	Pot Wiper
Pin 6	Radio Common
Pin 7	Nurse Call Interlock
Pin 8	Audio Transfer –
Pin 9	Audio Transfer +
Pin 10	Interlock +
Pin 11	Interlock –
Pin 12	Spare
Pin 13	Options 3 Common
Pin 14	Pot Low Common
Pin 15	Pot High Common
Pin 16	Nurse Answer Light +
Pin 17	Option 1 NO/NC
Pin 18	Option 1 Common
Pin 19	Nurse Call Light +
Pin 20	Option 2 NO/NC
Pin 21	Option 3 NO/NC
Pin 22	Option 3A NO/NC
Pin 23	Option 2A Common
Pin 24	Option 2A NO/NC
Pin 25	Nurse Call +
Pin 26	Nurse Call NO/NC
Pin 27	Room/Read Light Common
Pin 28	Nurse Call Light –
Pin 29	Nurse Answer Light –
Pin 30	Priority NO/NC
Pin 31	Priority Common
Pin 32	Option 3A Common
Pin 33	TV –
Pin 34	TV +
Pin 35	Speaker Low Common
Pin 36	Audio Shield
Pin 37	Radio NO/NC

BRAKE PEDAL REPLACEMENT

Required Tools:

5/16" Hex Allen Wrench Hammer Torque Wrench Punch Loctite 242 #2 Phillips Screwdriver

Procedure:

1. Unplug the power cord from the wall socket.

2. Using a #2 Phillips screwdriver, remove the three Phillips head screws holding both the head end and the foot end upper lift covers. If desired, hold the covers out of the way by using bungee cords (or the equivalent) to secure them to the litter top.

3. Using a 5/16" hex Allen wrench, remove the two bolts holding the brake pedal to the brake rod.

4. Using a hammer and punch, remove the roll pins holding the brake shaft crank to the brake rod on both the head and the foot end.

5. Push the brake rod through the frame until the brake pedal is clear. Remove the brake pedal.

6. Reverse the above steps to attach the new brake pedal.

NOTE

Use Loctite 242 when reinstalling the bolts and torque the bolts to 25 foot-pounds.


BRAKE SENSOR REPLACEMENT

Required Tools:

5/16" Socket Wrench

Side Cutters

5/16" Open End Wrench

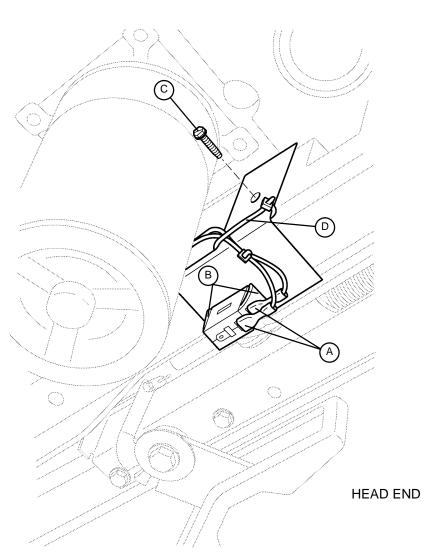

- 1. Unplug the power cord from the wall socket. Using a 5/16" socket wrench, remove the six bolts holding the lower lift cover to the base.
- 2. Remove the two cables (A) from the switch. (Note the terminals where the cables are connected so the cables will be reattached properly.)
- 3. Squeeze the switch retaining clips (B) and push up on the switch to remove it from the retaining bracket.
- 4. If the retaining bracket is bent or damaged, remove the screw (C) and the cable tie (D) holding the bracket to the frame and remove the bracket.
- 5. Reverse the above steps to install the new bracket and/or switch.


STEER WHEEL CABLE REPLACEMENT AND ADJUSTMENT

Required Tools:

5/16" Socket WrenchStandard Screwdriver(2) 3/16" Hex Socket Drivers


- (2) 5/16" Nut Drivers
- (2) 10 mm Open End Wrenches

Needle–Nose Vise Grip Pliers 3/8" Socket Wrench

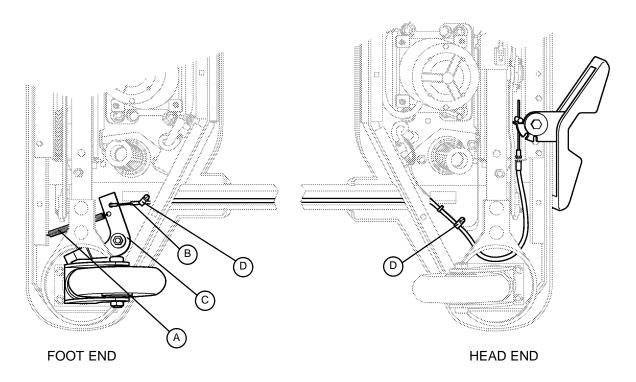

- 1. Unplug the power cord from the wall socket. Using a 5/16" socket wrench, remove the six bolts holding the lower lift cover to the base and remove the cover.
- 2. Remove the steer lock spring (A).
- 3. Using a 3/8" socket wrench, remove the cable clamp (D) from the foot end and remove the steer cable (B) from the steer latch lever (C).
- 4. Using two 10 mm open end wrenches, remove the two jam nuts securing the cable to the cable support bracket at the head end of the bed on the patient's right side.
- 5. Using a 5/16" nut driver, remove the cable clamp (D) holding the cable to the base at the head end of the bed.

STEER WHEEL CABLE REPLACEMENT AND ADJUSTMENT (CONTINUED)

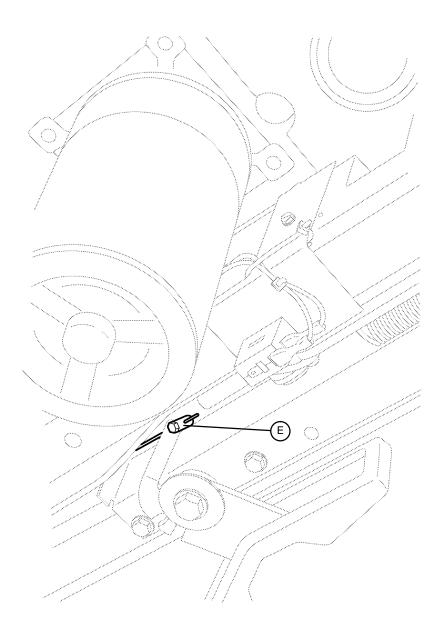


6. Using two 3/16" hex socket drivers, loosen the bolt in the cable retainer (E) and slide the retainer off the cable. Pull the cable out of the bed.

NOTE

If the cable sheathing is not damaged, pull the cable from the head end of the bed and remove it from the sheathing, leaving the sheathing on the bed. Push the new cable through the sheathing from the head end.

7. Reverse the above steps to attach the new cable.

STEER WHEEL CABLE REPLACEMENT AND ADJUSTMENT (CONTINUED)

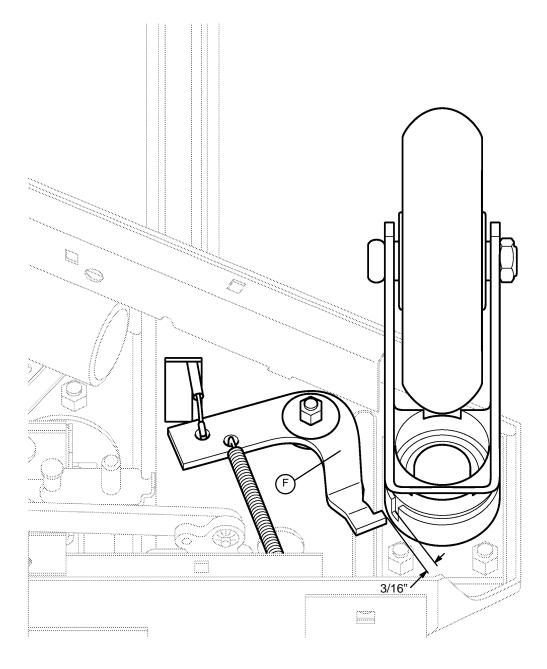

Adjustment Procedure:

If the steer wheel engages when the steer pedal is in the neutral or off position, slide the cable retainer toward the cable support bracket and tighten the retainer back down on the cable.

If the steer wheel does not hold properly when the pedal is in the steer position, slide the retainer away from the cable support bracket and tighten the retainer back down on the cable.

NOTE


The ideal off position for the steer latch lever (F) to the steer wheel is approximately 3/16" between the end of the lever and the caster horn.

CASTER REPLACEMENT

Required Tools:

Floor Jack #2 Phillips Screwdriver 1/2" Socket w/extension Bungee Cord (or equivalent)

Wooden Brace

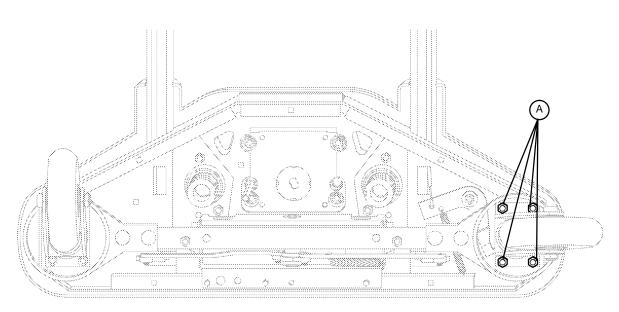

FOOT END – BOTTOM VIEW

Replacement Procedure:

- 1. Unplug the power cord from the wall socket.
- 2. Using a #2 Phillips screwdriver, remove the three screws holding the upper lift cover to the base.
- 3. Be sure the bed's brakes are on. Using a floor jack and wooden brace, lift the end of the bed with the defective caster off the floor a few inches.
- 4. Using a 1/2" socket with extension under the base, remove the four castle nuts (A) holding the caster assembly to the base.
- 5. Reverse the above procedure to install the replacement caster.


NOTE

The upper lift cover was raised to allow access to the four carriage bolts which need to be held when the four castle nuts are reinstalled.

LIFT MOTOR AND CAPACITOR REMOVAL AND REPLACEMENT

Required Tools:

3/8" Socket Wrench 7/16" Open End Wrench

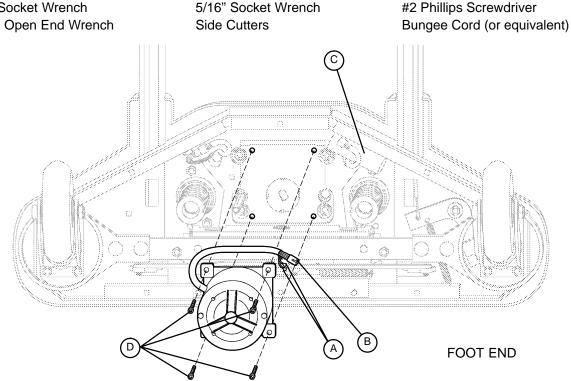

Procedure:

- 1. Unplug the power cord from the wall socket. Using a 5/16" socket wrench, remove the six bolts holding the lower lift cover to the base and remove the cover.
- 2. Using a #2 Phillips screwdriver, remove the three screws holding the upper lift cover to the base.
- 3. Disconnect the two connectors (A) at the motor capacitor.
- 4. Disconnect the white connector (B) from the power cord.
- 5. Using side cutters, cut the cable ties holding the capacitor (C) to the base and remove the capacitor.
- 6. Using a 3/8" socket wrench, remove the four screws (D) holding the motor assembly in the lift housing and remove the motor assembly.
- 7. Use the access holes in the lift housing to view the alignment of the new motor drive shaft to the motor coupler. Lift the motor into place and secure it with the bolts removed in step 7. Hold the nuts that the bolts are being turned into through the access holes until the threads of the bolts are started.
- 8. Using cable ties, reinstall the new capacitor to the base.

NOTE

The drive shaft on the new motor probably will have to be turned to be aligned with the coupler. Use a 7/16" open end wrench to turn the drive shaft of the motor.

Reattach the three connectors, return all wiring to its original position and reinstall all panels and covers. 9.

NOTE

The procedure for lift motor and capacitor removal and replacement is the same for both ends of the bed.


LIFT MOTOR ISOLATION PLATE REPLACEMENT

Required Tools:

#2 Phillips Screwdriver3/8" Socket WrenchNeedle–Nose Pliers3/16" Hex Allen Socket (w/6" ext.)

1/4" Nut Driver3/16" Punch5/16" Nut Driver5/16" Socket Wrench

3/8" Open End Wrench Hammer 7/16" Open End Wrench

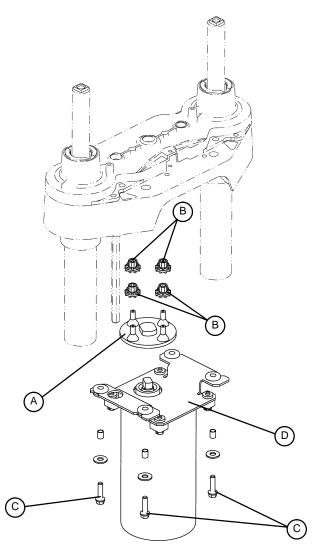

- 1. Unplug the power cord from the wall socket.
- 2. Using a #2 Phillips screwdriver, remove the three screws holding the upper lift cover to the base.
- 3. Using a 5/16" socket wrench, remove the six bolts holding the lower lift cover to the base and remove the cover.

LIFT MOTOR ISOLATION PLATE REPLACEMENT (CONTINUED)

4. Using a 3/16" hex Allen socket with 6" extension, remove the four bolts holding the lift motor to the base and carefully lower the motor to the floor.

- 5. Remove the motor coupler (A) and bushings (B) through the access holes.
- 6. Using a 3/16" punch and hammer, remove the top roll pin on the manual drive hex shaft coupling and set the shaft and coupler assembly aside.
- 7. Using a 3/8" open end wrench, remove the four bolts, washers, and spacers (C) from the lift housing and remove the isolation plate (D).
- 8. Rotate the isolation plate toward the access opening and tilt it up and out of the opening.
- 9. Reverse the above steps to install the new isolation plate and reattach all components to the bed.

NOTE

When reinstalling the lift motor, use the access holes in the lift housing to view the alignment of the motor drive shaft to the motor coupler. The drive shaft on the motor probably will have to be turned to be aligned with the coupler. Use a 7/16" open end wrench to turn the drive shaft of the motor.

LIFT HOUSING REMOVAL AND REPLACEMENT

Required Tools:

Standard ScrewdriverPhillips Screwdriver5/16" Socket Wrench9/16" Socket Wrench3/8" Open End WrenchSmall Standard ScrewdriverSide Cutters5/16" Nut Driver3/16" Hex Allen Socket (w/ approx. 6" extension)


Bungee Cord (or equivalent) Needle–Nose Vise Grip Pliers 5/16" Open End Wrench 7/32" Hex Allen Socket Wrench Sawhorses (or equivalent)

Procedure:


- 1. Unplug the power cord from the wall socket.
- 2. Using a 5/16" socket wrench, remove the six bolts holding the lower lift cover to the base and remove the cover.
- 3. Using a #2 Phillips screwdriver, remove the three screws holding the upper lift cover to the base.
- 4. Using a #2 Phillips screwdriver, remove the four screws holding the litter access panel at the end of the bed needing service.
- 5. Remove the lift motor and capacitor (refer to procedure on page 55).
- 6. Remove lift potentiometer (refer to procedure on page 60).
- 7. Using a 5/16" open end wrench, remove the cable clamps holding the power and sensor coil cords on top of the lift housing assembly. Cut the cable ties and disconnect the coil cords from under the lift housing. The power and sensor coil cords are now free of the lift housing assembly. Drape them up out of the way.
- 8. Using a 7/32" hex Allen socket, remove the two screws (A) holding the lift screws to the header crossbar plate.

CAUTION

The bed litter retracts on rollers. Secure it to prevent it from rolling while the procedure is being done.

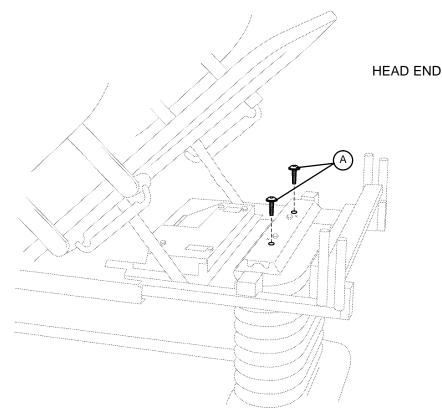

LIFT HOUSING REMOVAL AND REPLACEMENT (CONTINUED)

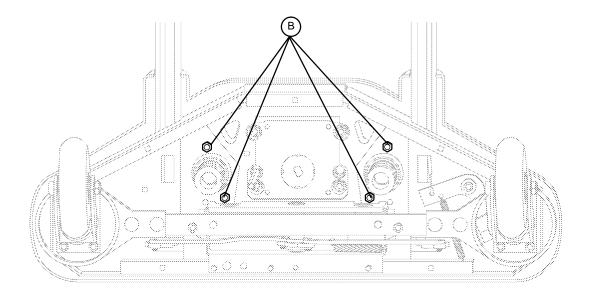

FOOT END – BOTTOM VIEW

- 9. Lift the litter top up and support it about 6" above the lift screws with sawhorses or the equivalent.
- 10. Under the base, using a 9/16" socket, remove the four nuts (B) holding the lift housing to the base.
- 11. Lift up and out on the lift housing assembly to remove it from the base.

CAUTION

See pages 63 and 64 for proper reattachment procedure for power and sensor coil cords.

12. Reverse the above steps to reinstall the lift housing assembly after service is completed.

NOTE


The procedure for lift housing removal and replacement is the same for both ends of the bed.

LIFT POTENTIOMETER REPLACEMENT AND ADJUSTMENT

Required Tools:

Small Standard Screwdriver 5/16" Socket Wrench

#2 Phillips Screwdriver 3/8" Open End Wrench Bungee Cord (or equivalent) Side Cutters

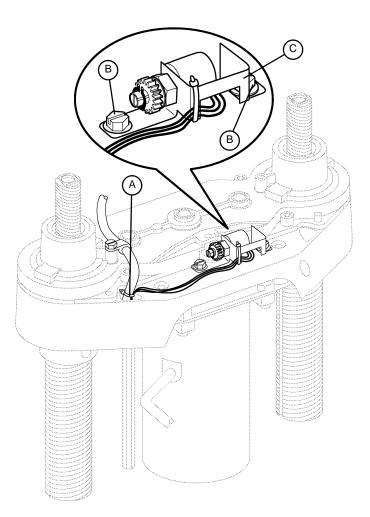

- 1. Unplug the bed power cord from the wall socket.
- 2. Using a 5/16" socket wrench, remove the six bolts holding the lower lift cover to the base and remove the cover.
- 3. Using a #2 Phillips screwdriver, remove the three screws holding the upper lift cover to the base.
- 4. Using side cutters, cut the cable tie (A) holding the pot cable to the coil cord.
- 5. Unplug the pot cable from the sensor coil cord. If replacing a pot at the head end of the bed, unplug the cables attached to the brake sensor switch.
- 6. Pull the pot cable up through the base.
- 7. Using a 3/8" open end wrench, remove the two bolts (B) holding the pot housing (C) to the lift housing.

LIFT POTENTIOMETER REPLACEMENT AND ADJUSTMENT (CONTINUED)

- 8. Lift up and out on the pot housing assembly to remove it from the lift housing.
- 9. Before installing the new pot on the bed, turn it clockwise until it stops. Turn it back counterclockwise two full (360°) revolutions. This allows a "window" position for proper upper and lower limits.
- 10. Reverse steps 4–8 to install the new pot and pot housing assembly.

NOTE

Be sure to maintain the pot position while installing.

11. After installing the new pot, the "burn-in" procedure below must be followed.

LIFT POTENTIOMETER "BURN-IN" PROCEDURE


- 1. Unplug the power cord from the wall socket.
- 2. On the foot board control panel, hold down the Lights, Fowler Lock Out, and Up/Down Lockout buttons simultaneously.
- 3. While holding down the above three buttons, plug the power cord into the wall socket. The Lights button on the foot board control panel should flash, indicating the bed is in the diagnostics mode.
- 4. Run both ends of the bed full down to a "hard stop".
- 5. Hold down the Bed Motion Lock button until the light flashes.
- 6. Release the button and unplug the power cord from the wall socket.
- 7. Plug the power cord back in to the wall socket. Run the bed to full up, then full down to verify the bed limits.
- 8. The distance between the floor and the top of the litter seat section (without a mattress) should be 16^{-16} $16 = 3/8^{-16}$ with the litter fully down and $29 = 1/4^{-16} 29 = 7/8^{-16}$ with the litter fully up.

NOTE

These values are for beds equipped with 6 inch casters. Add two inches to both measurements for beds equipped with 8 inch casters.


LIFT MOTOR COUPLER REPLACEMENT

Required Tools:

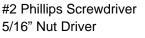

Bungee Cord (or equivalent) 1/2" Socket w/extension

5/16" Socket Wrench

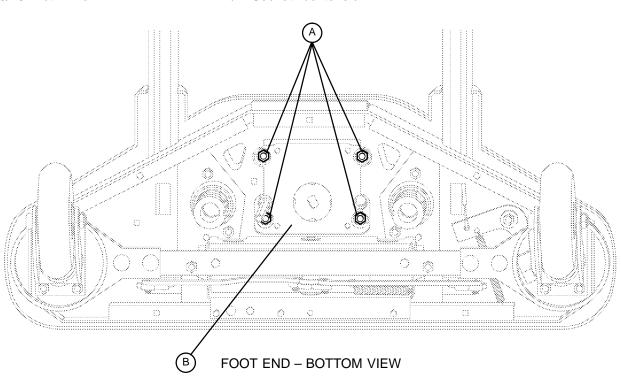

- 1. Unplug the power cord from the wall socket.
- 2. Using a #2 Phillips screwdriver, remove the three screws holding the upper lift cover to the base.
- 3. Using a 5/16" socket wrench, remove the six bolts holding the lower lift cover to the base and remove the cover.
- 4. Using a 3/8" socket with an extension, remove the four bolts (A) holding the isolation plate (B) to the lift housing and lower the lift motor and isolation plate assembly to allow access to the coupler. Take care not to drop the isolation plate onto the base.
- 5. The motor coupler can now be removed from the lift housing through the access hole.
- 6. Reverse the above steps to install the new motor coupler and bushings.

POWER AND SENSOR COIL CORD REPLACEMENT

Required Tools:

#2 Phillips ScrewdriverBungee Cord (or equivalent)5/16" Socket Wrench

Side Cutters 5/16" Nut Driver

Procedure:

1. Unplug the bed power cord from the wall socket.

2. Using a 5/16" socket wrench, remove the six bolts holding the lower lift cover to the base and remove the cover.

- 3. Using a #2 Phillips screwdriver, remove the three screws holding the upper lift cover to the base.
- 4. Using a #2 Phillips screwdriver, remove the four screws holding the litter access panels at the end of the bed needing service.
- 5. Using side cutters, cut the cable ties holding the power and sensor coil cords to the base. Remove the ground wire coming from the sensor cord that is attached to the base (note the star washer arrangement).
- 6. Disconnect the cables going to the motor and the lift potentiometer (at the head end, the sensor cord is also attached to the brake switch sensor).
- 7. Pull both cords up through the frame of the bed and the lift housing.
- 8. Using a 5/16" nut driver, remove the two screws (A) holding the cable clamps* to the top of the lift housing.
- 9. Using a 5/16" nut driver, remove the two screws (B) securing the cable clamps* to the underside of the header crossbar assembly.
- 10. Pull both coil cords up through the header crossbar assembly.
- 11. Disconnect the power and sensor coil cords from the connectors.
- 12. The cords should now be completely removed from the bed. Reverse the above steps to install the new power and sensor cords*.

CAUTION

*When the power and sensor coil cords are being replaced, secure the cable clamps to the cords at the first coil on the top and on the bottom to assure there is not too much slack in the cords between the top of the lift housing assembly and the bottom of the header crossbar. Be sure the clamps are fastened at exactly the correct angle, as shown by the arrows in the illustration. Arrange the cords exactly as shown in the illustration (left in front of right). If this is not done correctly, damage to the cords will result.

В (в) А А

POWER AND SENSOR COIL CORD REPLACEMENT ILLUSTRATION

VIEW FROM CENTER OF BED

MOTION INTERRUPT SWITCH REPLACEMENT

Required Tools:

#2 Phillips Screwdriver 5/16" N

5/16" Nut Driver

3/8" Socket Wrench OR T27 Torx

NOTE

Because of fastener changes, it will be necessary to use either a 3/8" socket wrench or a T27 Torx to remove the bolts holding the ground straps to the motion interrupt pan and a Phillips screwdriver or a T27 Torx to remove the bolts securing the litter access panels.

Procedure:

- 1. Unplug the power cord from the wall receptacle.
- 2. Using a 3/8" socket wrench or a T27 Torx, remove the two bolts holding the ground straps to the motion interrupt pan at the foot end of the bed.
- 3. Using a 5/16" nut driver, remove the two bolts holding the motion interrupt pan to the litter and remove the motion interrupt pan.
- 4. Using a #2 Phillips screwdriver or a T27 Torx, remove the four screws securing the litter access panel at the end of the bed needing service and remove the panel.
- 5. Disconnect the two wires at the switch being replaced.

NOTE

Take note of the terminals being used on the switch so the wires will be attached properly to the new switch.


- 6. Squeeze the retention clips on the switch and push down on the switch to remove it.
- 7. Reverse the above steps to install the new switch.
- 8. Activate the motion stop system to assure it is functioning properly: press and hold down the BED DOWN button. While the bed is lowering, lift up on each corner of the motion interrupt pan individually and ensure the downward motion stops each time. Release the pan and allow the bed's downward motion to continue.

KNEE MOTOR REMOVAL AND REPLACEMENT

Required Tools:

3/8" Nut Driver Wire Cutters T27 Torx Socket Wrench Needle Nosed Pliers 3/8" Socket

- 1. Electrically run the litter fully up. Unplug the bed power cord from the wall receptacle.
- 2. Using a 3/8" nut driver, remove the two bolts holding the ground straps to the motion interrupt pan.
- 3. Using the 3/8" nut driver, remove the two bolts holding the motion interrupt pan to the litter frame and remove the pan.
- 4. Using needle nose pliers, remove the Rue pin (A) from the end of the drive tube.
- 5. Remove the two wires from the auto contour micro switch (B) mounted next to the drive tube.
- 6. Fold the Gatch section back toward the head end of the bed. Using a T27 Torx, remove the screw holding the mid–litter cover to the litter frame and remove the cover.
- 7. Using wire cutters, remove the cable ties. Disconnect the motor and capacitor wires.
- 8. Using needle nosed pliers, remove the Rue pin (C) from the motor bracket and lift the motor out.
- 9. Using a 3/8" socket wrench, remove the four bolts holding the motor bracket to the old motor. Remove the bracket and install it on the replacement motor using the same four bolts.
- 10. Install the replacement motor into the bed. **Do not** hook up the drive tube to the Gatch section yet.
- 11. Plug the bed power cord into a properly grounded wall receptacle. Using the button on the foot board or siderail, run the Gatch motor down until it stops.
- 12. Reinstall the mid-litter cover.
- 13. Fold the Gatch section back down. From underneath the bed, manually turn the drive tube in or out until it lines up with the hole in the Gatch section. Reinstall the pin, the wires for the auto contour switch, and the motion interrupt pan.
- 14. Test the bed for proper operation before returning it to service.

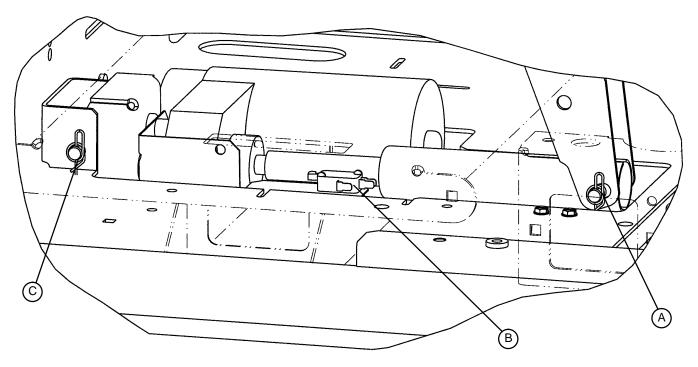


HEAD MOTOR CAM AND CAM GUIDE REPLACEMENT

Required Tools:

5/16" Nut Driver 7/16" Open End Wrench 7/16" Nut Driver 3/8" Socket Wrench or T27 Torx #2 Phillips Screwdriver

NOTE

Because of fastener changes, it will be necessary to use either a 3/8" socket wrench or a T27 Torx to remove the bolts holding the ground straps to the motion interrupt pan and a Phillips screwdriver or a T27 Torx to remove the bolts securing the litter access panels.

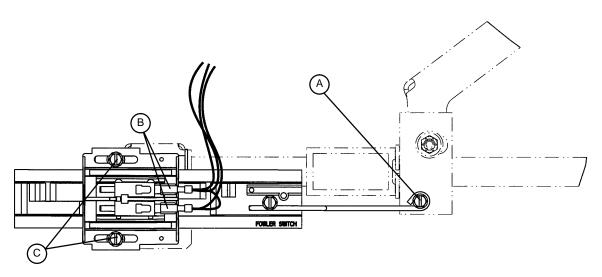

- 1. Unplug the power cord from the wall socket.
- 2. Using a 3/8" socket wrench or a T27 Torx, remove the two bolts holding the ground straps to the motion interrupt pan.
- 3. Using a 5/16" nut driver, remove the two bolts holding the motion interrupt pan to the litter and remove the pan.
- 4. Using a #2 Phillips screwdriver or a T27 Torx, remove the four screws holding the litter access cover to the litter at the foot end of the bed and remove the cover.
- 5. Using a 5/16" nut driver, remove the screw (A) holding the cam retention wire to the ball nut assembly.
- 6. Disconnect the cables (B) from the micro switches.
- 7. Using a 5/16" nut driver, remove the two screws (C) holding the cam and cam guide to the support bracket and remove the cam and cam guide with the attached micro switches from the bed.
- 8. Reverse the above steps to install the replacement cam and cam guide/micro switch assembly.
- 9. Refer to page 68 for "Limit Setting Head Motor".

LIMIT SETTING – HEAD MOTOR

Required Tools:


#2 Phillips Screwdriver 7/16" Socket

5/16" Nut Driver 3/8" Socket Wrench or T27 Torx

7/16" Open End Wrench

NOTE

Because of fastener changes, it will be necessary to use either a 3/8" socket wrench or a T27 Torx to remove the bolts holding the ground straps to the motion interrupt pan and a Phillips screwdriver or a T27 Torx to remove the bolts securing the litter access panels.

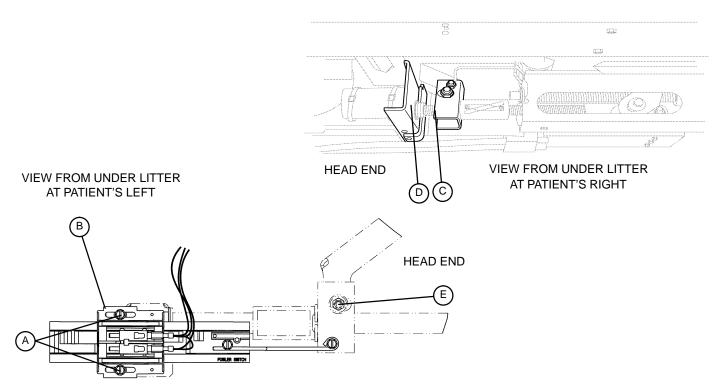

- 1. Using a #2 Phillips screwdriver or a T27 Torx, remove the ten screws holding the foot and midsection litter access covers to the litter and remove the covers.
- 2. Using a 3/8" socket wrench or a T27 Torx, remove the two bolts holding the ground straps to the motion interrupt pan.
- 3. Run the head section down electrically until flat (0°) or use the CPR release handle to fully lower the head.
- 4. Lightly tap the Fowler down button until the bolt and bushing (E) are centered in the slot.
- 5. Unplug the power cord from the wall socket.
- 6. Using a 5/16" nut driver, loosen the two screws (A) on the cam (B).

LIMIT SETTING - HEAD MOTOR (CONTINUED)

7. Slide the cam toward the head of the bed just until the microswitch is activated (the switch will make a clicking noise).

NOTE

Check that the cam "bump" is on the "head" side of the switch actuator.

8. Tighten the two screws (A).

CAUTION

Be sure the head section stops when it is fully up and the ball nut bracket (C) is approximately 3/4" from panel (D). If not, damage to the unit could result.

- 9. Ensure the head section will lower to flat (0°) before returning the bed to service. Verify the head section lowers onto the rubber rests and the motor stops running.
- 10. Re-install the litter access panels removed in step 1.

HEAD MOTOR REMOVAL AND REPLACEMENT

Required Tools:

Side Cutters	5/16" Nut Driver	7/16" Socket Wrench
3/8" Socket Wrench	#2 Phillips Screwdriver	T27 Torx

NOTE

Because of fastener changes, it will be necessary to use either a Phillips screwdriver or a T27 Torx to remove the bolts securing the litter access panel.

Procedure:

WARNING

The head motor is not grounded and may be an electrical hazard. Use caution when working around motors.

- 1. Run the head section (Fowler) up to approximately 30° 35° for better access to the motor.
- 2. Unplug the power cord from the wall socket.
- 3. Using a #2 Phillips screwdriver or a T27 Torx, remove the four screws holding the head end litter access cover to the litter and remove the cover.
- 4. Using side cutters, cut the cable tie holding the CPR release cable to the litter (to allow clearance for lifting the motor out of the litter).

HEAD MOTOR REMOVAL AND REPLACEMENT (CONTINUED)

5. Unplug the head motor cable (A) from the power board.

WARNING

Support the head section with a wooden brace (2x4 or equivalent) before continuing or personal injury could result.

- 6. Using a 5/16" nut driver, remove the two screws and washers (B) holding the motor mount to the litter.
- 7. Unplug the two cables (C) at the head motor capacitor.
- 8. To remove the head motor from the litter, push the motor toward the foot end and lift straight up on the motor about 1". Pull the motor toward the head end and lift it straight up and out.

NOTE


The replacement motor will come with the attached CPR clutch and clutch bushings.

9. Using a 7/16" socket, remove the two bolts holding the motor to the mounting bracket.

NOTE

The mounting bracket will be re-used with the replacement motor.

10. Reverse the above steps to install the new motor assembly and reattach all litter panels. Be sure to attach new cable ties wherever they were removed.

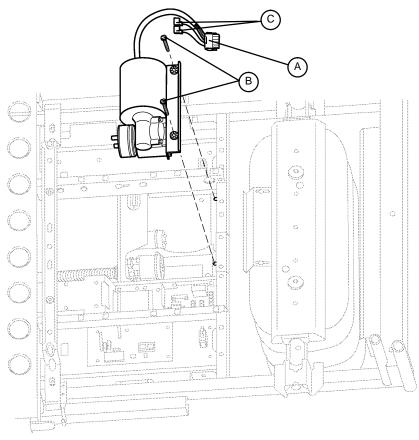


TOP VIEW

HEAD MOTOR DRIVE ISOLATOR AND CPR DECOUPLER REMOVAL AND REPLACEMENT

Required Tools:

5/16" Nut Driver 7/16" Socket Wrench #2 Phillips Screwdriver Wire Cutter

3/8" Socket Wrench

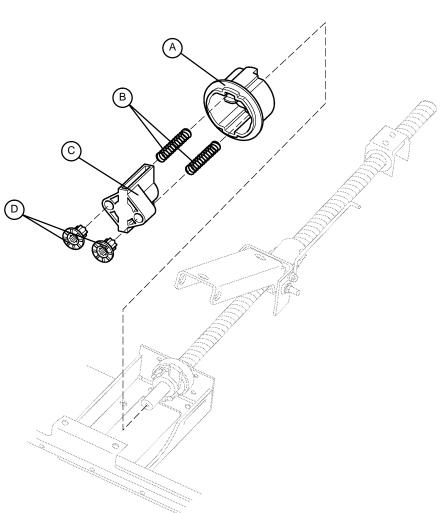

Procedure:

- 1. Refer to page 69 for the "Head Motor Removal and Replacement" procedure to access the isolator, springs and decoupler.
- 2. Pull the drive isolator (C) with the bushings (D), springs (B) and CPR decoupler (A) off the drive screw and remove it from the litter.

NOTE

These parts are spring-loaded. Use caution when removing them so they won't drop to the floor.

3. Reverse the above steps to install the new isolator and decoupler.


DRIVE SCREW AND BALL NUT REPLACEMENT - HEAD MOTOR

Required Tools:

1/4" Nut Driver Needle–Nose Pliers 5/16" Nut Driver 7/16" Open End Wrench (2) 7/16" Socket Wrench3/8" Socket Wrench or T27 Torx

NOTE

Because of fastener changes, it will be necessary to use either a 3/8" socket wrench or a T27 Torx to remove the bolts holding the ground straps to the motion interrupt pan.

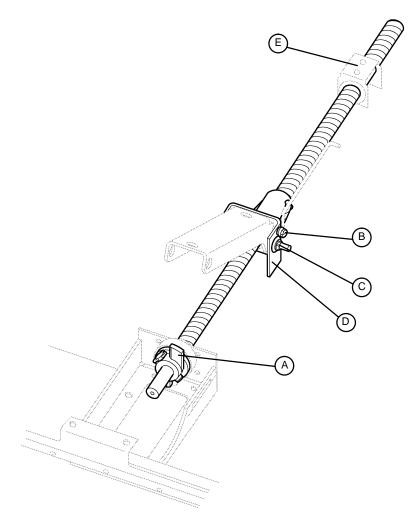

Procedure:

- 1. Unplug the power cord from the wall socket.
- 2. Using a 3/8" socket wrench or a T27 Torx, remove the two bolts holding the ground straps to the motion interrupt pan.
- 3. Using a 5/16" nut driver, remove the two bolts holding the motion interrupt pan to the litter and remove the pan.
- 4. Remove the head motor (see page 69) for access to the screw and ball nut.
- 5. Remove the head motor drive isolator and CPR decoupler (see page 71).
- 6. Using needle-nose pliers, remove the cotter pin holding the clevis pin to the drive screw "wing" (A).

DRIVE SCREW AND BALL NUT REPLACEMENT - HEAD MOTOR (CONTINUED)

- 7. Remove the clevis pin and slide the wing and tapered roller bearing off the drive screw.
- 8. Using a 5/16" nut driver underneath the litter, remove the screw holding the cam retention wire (B) to the ball nut assembly.
- 9. Slide the cam and retention wire as far toward the foot end of the bed as possible to allow proper clearance for removal of the drive screw.
- 10. Remove the hinge pin from the Fowler and main link.
- 11. To prevent the ball nut from coming off the drive screw, cable tie both ends of the ball nut to the drive screw.
- 12. Pull the drive screw toward the foot of the bed and remove the head end of the drive screw and ball nut assembly from the support (D). Push the drive screw toward the head end of the bed and remove the foot end of the screw from the glide bushing support (E).

NOTE

Do not allow the ball nut assembly to come off the drive screw. The ball nut will be shipped attached to the drive screw and retained in position during shipment. Remove the restraining straps **after** installing the drive screw and nut.

- 13. Remove the thrust washers and bearing from the head end of the drive screw and set them aside.
- 14. Using two 7/16" open end wrenches, remove the bolt, nut and bushings (C) supporting the ball nut assembly on the Fowler link.
- 15. Reverse the above steps to install the replacement drive screw and ball nut assembly.

NOTE

Apply grease to all bearing areas.

CAUTION

Assure the head motor stops properly at both upper and lower electric limits. If it does not, refer to page 68 for "Limit Setting – Head Motor".

WEIGH SYSTEM DIAGNOSTIC PROCEDURE

Diagnostic Mode Functions:

- 1. Calibrate Scale: This may be required in the field if the scale CPU is replaced.
- 2. Init. to Defaults: This may be required in the field when replacing the scale CPU board.
- 3. **Display Corner:** This function displays the individual corner weights for each load cell assembly and can be used to isolate a defective load cell.
- 4. Exit Diagnostics: Changes made in the diagnostic mode (with the exception of Calibrate Scale) **must** be saved in permanent memory using this function. Switching off power without saving will reset all variables to their previous values.

Diagnostic Mode:

NOTE

It requires **two people** to enable the diagnostic mode for the scale system.

- 1. To enter diagnostic mode, unplug the bed's power cord from the wall socket.
- 2. Press and hold down the LBS/KGS key.
- 3. While still holding the LBS/KGS key, plug the bed's power cord into the wall socket.
- 4. After two seconds, release the LBS/KGS key. The scale monitor should read CALIBRATE SCALE. The diagnostic mode is now active.

Special Key Functions in the Diagnostic Mode:

NOTE

By pressing the LBS/KGS key, the diagnostic mode functions can be scrolled. To enter the selected mode, press the SCALE ON key.

- 1. The four keys listed in the group below function as POSITION keys corresponding with the four corners of the bed's litter. Whenever the scale monitor displays PICK CORNER NOW, press one of these keys to select the load cell assembly at the desired corner.
- A. ZERO = head end, patient's right side
- B. MINUS = foot end, patient's left side
- C. DELTA = head end, patient's left side
- D. PLUS = foot end, patient's right side

WEIGH SYSTEM DIAGNOSTIC PROCEDURE (CONTINUED)

Displaying Individual Load Cell Outputs:

A defective load cell can be detected by entering diagnostics and displaying individual load cell outputs.

- 1. Enter the diagnostic mode. The scale monitor will display CALIBRATE when the diagnostic mode is active.
- 2. Press and release LBS/KGS until the scale monitor displays DISPLAY CORNER.
- 3. Press and release SCALE ON. The scale monitor should display PICK CORNER NOW.
- 4. Press and release the position key that corresponds with the load cell to be checked. The scale monitor should display X/X=NNN.N. "X/X" represents the initials of the selected corner, i.e. H/R will be displayed for the patient's head end, right side. "NNN.N" represents the actual weight load on the load cell.
- 5. Repeat step four for each corner. Head end weight readings will normally be lower than foot end weights. Weight readings should be constant. A drifting 000.0 or 999.9 weight, or a reading that does not change when weight is applied to that corner of the bed indicates a problem with the selected load cell assembly or load cell cable.
- 6. When all the load cell outputs have been checked, press and release SCALE ON. To exit diagnostics, unplug the bed's power cord from the wall socket.

Scale Calibration:

NOTE

It requires **two people** to enable the diagnostic mode for the scale system.

- 1. Zero the empty bed. Place a known weight on the center of the bed; the heavier the better and no less than 100 pounds. The displayed weight should be within 1% of the actual weight.
- 2. If the displayed weight is not accurate, use the following procedure to calibrate the scale:

A. Remove the weight from the bed and place the scale CPU in diagnostics mode (see diagnostic mode section on previous page).

- B. The scale monitor should display CALIBRATE SCALE.
- C. Press and hold SCALE ON. Zero the bed, following the displayed instructions. When the bed is zeroed, the scale monitor should display REF X100=<2>000.0.
- D. The displayed number must now be changed to match the known weight times 100. For example, if you weighed yourself on a known accurate scale and you weigh 192.4 pounds, you would change the REF number to equal 19240. Change the REF number using the LBS/KGS key to position the brackets (<>) around the digit needing to be changed and the "+" key to scroll through the bracketed digit.
- E. After the number is corrected, press and release SCALE ON. The scale should display ADD LBS, HIT ON.
- F. Place weight used in step one on the center of the bed and press and hold SCALE ON. The scale monitor should display RELEASE TO CAL.
- G. Release SCALE ON. The scale monitor should display DO NOT TOUCH BED. Don't touch the bed. When calibration is complete, the scale monitor should display CALIBRATE SCALE.

H. Calibration is now complete, press LBS/KG until EXIT DIAGNOSTICS appears. Press SCALE ON and the display will read WEIGHING. Re–zero the empty bed. Check the scale accuracy with the weight used previously.

LOAD CELL REPLACEMENT

Required Tools:


#2 Phillips Screwdriver or T27 Torx 5/16" Nut Driver

Side Cutters

1/4" Hex Allen Wrench

NOTE

Because of fastener changes, it will be necessary to use either a Phillips screwdriver or a T27 Torx to remove the bolts securing the litter access panel.

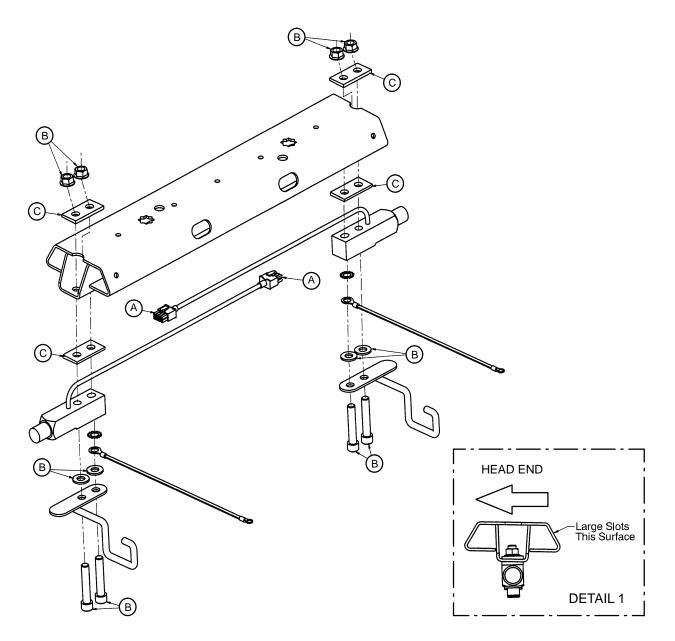

Procedure:

- 1. Unplug the bed power cord from the wall socket.
- 2. Using a #2 Phillips screwdriver or a T27 Torx, remove the four screws securing the litter access cover at the end of the bed needing service and remove the cover.

LOAD CELL REPLACEMENT (CONTINUED)

- 3. Using a 5/16" nut driver, remove the bolt holding the load cell cable connector cover and remove the cover.
- 4. Support the corner of the litter where the load cell is being removed.
- 5. Unplug the load cell connector (A) from the load cell cable. Using side cutters, cut the cable ties.
- 6. Using a 1/4" hex Allen wrench, remove the two screws, washers, and nuts (B) holding the load cell to the bed.
- 7. To remove a load cell at the head end of the bed, slide the load cell toward the center of the bed to clear the bushing that supports the load cell. Remove the load cell by lifting it up and out. To remove a load cell at the foot end of the bed, slide the load cell with the roller bushing toward the center of the bed to clear the litter frame. Remove the load cell and the roller bushing by lifting them up and out.

NOTE

If your bed is equipped with a scale system, there are two tapered shims (C) for each load cell. Refer to detail 1 on page 76 as reference for proper installation of the shims. Proper installation of these shims is very important for scale accuracy.

8. Reverse the above procedure to install the new load cell.

CAUTION

Be sure the load cell's strain gauge is facing up when installing a new load cell or the weigh system will not work properly.

9. Recalibrate the weigh system (see page 75).

POWER BOARD REPLACEMENT

Required Tools:

1/4" Nut Driver#2 Phillips Screwdriver or T27 TorxNeedle-Nose Pliers

NOTE

Because of fastener changes, it will be necessary to use either a Phillips screwdriver or a T27 Torx to remove the bolts securing the litter access panel.

Procedure:

- 1. Properly ground yourself (see page 29 for static discharge precautions).
- 2. Using a #2 Phillips screwdriver or a T27 Torx, remove the four screws holding the head end litter access cover and remove the cover.
- 3. Disconnect all cables at the power board. Note the locations of the cables so they will be reconnected properly to the replacement power board.
- 4. To remove the board from the litter standoffs, use a 1/4" nut driver to remove the six screws holding the board to the litter.
- 5. Install the new board and reconnect all cables.
- 6. Reinstall the litter access panel and plug the bed power cord into a properly grounded wall receptacle.
- 7. Test the bed for accurate operation before returning it to service.

CPU BOARD REPLACEMENT

Required Tools:

#2 Phillips Screwdriver or T27 Torx

NOTE

Because of fastener changes, it will be necessary to use either a Phillips screwdriver or a T27 Torx to remove the bolts securing the litter access panel.

Procedure:

- 1. Properly ground yourself (see page 29 for static discharge precautions).
- 2. Using a #2 Phillips screwdriver or a T27 Torx, remove the four screws holding the head end litter access cover and remove the cover.
- 3. Disconnect all cables at the litter CPU board. Note the locations of the cables so they will be connected properly to the replacement litter CPU.
- 4. To remove the board from the litter standoffs, pull out on the retention clips and lift upward on the board.
- 5. Install the new board and reconnect all cables.
- 6. Reinstall the litter access panel and plug the bed power cord into a properly grounded wall receptacle.
- 7. When a new CPU board is installed, a "burn-in" procedure must be done (see page 78). If your bed is equipped with a scale system, scale calibration must also be done (see page 75).
- 8. Test bed operations before returning the bed to service.

MAIN CPU BOARD "BURN-IN" PROCEDURE

When the main CPU board has been changed, the lift potentiometers must be programmed into the CPU.

Lift Potentiometer Programming

- 1. Unplug the power cord from the wall receptacle.
- 2. At the foot board, push and hold down the Siderail Lights button, Fowler Lockout button, and Bed Up/Down button.
- 3. While still holding down on the above three buttons, plug the power cord into the wall receptacle. The siderail "light" LED will blink.
- 4. Release the buttons. Run the bed all the way down until it stops.
- 5. Push and hold down the Bed Motion Lock button until the light flashes.
- 6. Unplug the power cord from the wall socket.
- 7. Plug the power cord back into the wall receptacle and run the bed up and down to verify the bed limits.


LITTER GREASE POINTS

- 1. All litter pivot joint bushings "Syntech" synthetic grease.
- 2. All Gatch and Fowler pivot link bushings "Syntech" synthetic grease.
- 3. Gatch drive screw and Gatch pan slide surface "Syntech" synthetic grease.
- 4. Fowler drive screw "Mobil 28" grease.

HEAD AND FOOT SIDERAIL COVER REMOVAL

Required Tools:

#2 Phillips Screwdriver

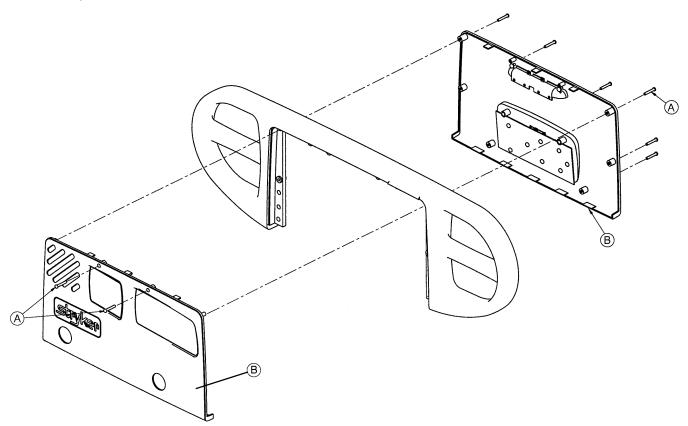

Removal Procedure:

- 1. Unplug the power cord from the wall receptacle.
- 2. Using a #2 Phillips screwdriver, remove the 8 phillips screws (A) holding the covers (B) to the siderail.

CAUTION

There are two cables connecting the outside cover to the head end siderail. Be careful not to pull on them when removing the cover.

- 3. Remove the cables from the siderail. Make note of the proper location for the cables.
- 4. Reverse the above steps to reattach the cover.

CAUTION

Do not snag the cables when installing the siderail cover.

NOTE


Follow the same procedure for siderail cover removal for the foot end rails.

HEAD AND FOOT MOLDED SIDERAIL REPLACEMENT

Required Tools:

#2 Phillips Screwdriver

3/8" Nut Driver

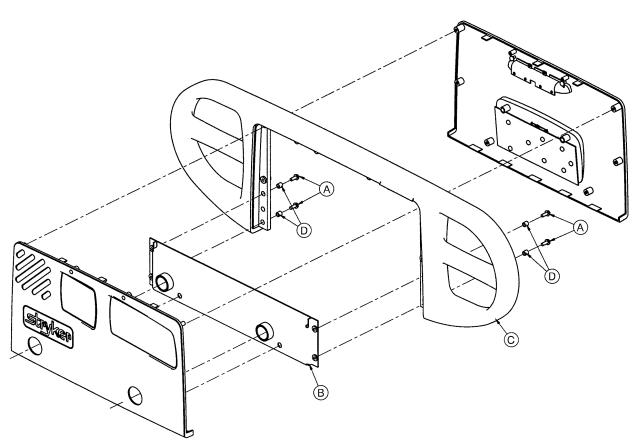

Procedure:

- 1. Unplug the bed power cord from the wall socket.
- 2. Remove the siderail cover (see page 79).
- 3. Using a 3/8" nut driver, remove the four screws (A) holding the molded rail (C) to the siderail support assembly (B).

NOTE

Note the location of the spacers (D) for re-assembly purposes.

- 4. Pull up on the molded rail (C) to remove it from the siderail assembly.
- 5. Reverse the above steps to install the new molded rail.

HEAD END SIDERAIL CABLE REPLACEMENT


Required Tools:

#2 Phillips Screwdriver or T27 Torx

Side Cutters

NOTE

Because of fastener changes, it will be necessary to use either a Phillips screwdriver or a T27 Torx to remove the bolts securing the litter access cover.

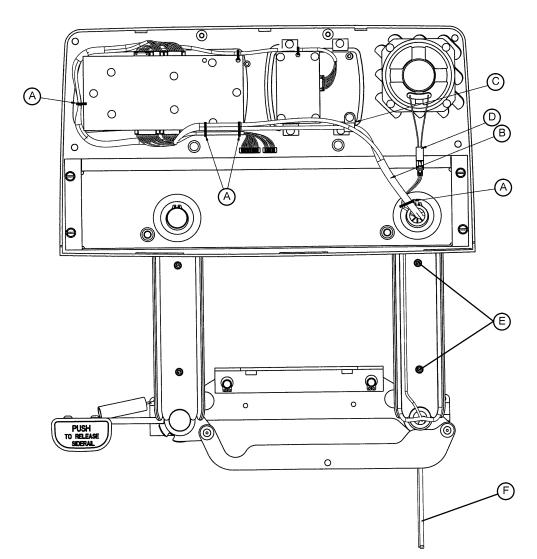

Procedure:

- 1. Run the head section fully up.
- 2. Unplug the bed power cord from the wall socket.
- 3. Using a #2 Phillips screwdriver or a T27 Torx, remove the four screws securing the head end litter access cover to the litter and remove the cover.
- 4. Using a #2 Phillips screwdriver, remove the eight screws securing the siderail cover and remove the cover.
- 5. Put the siderail in the down position.

HEAD END SIDERAIL CABLE REPLACEMENT (CONTINUED)

- 6. Using a Phillips screwdriver, remove the two screws (E) holding the rear siderail pivot arm cover to the pivot arm. Remove the cover to expose the siderail cables.
- 7. Using side cutters, clip the cable ties (A) holding the cables together.
- 8. Using a #2 Phillips screwdriver, remove the cable clamp (C) from the siderail.
- 9. Disconnect cable (B) from the circuit board and cable (D) from the speaker.

NOTE

The speaker and nurse call are optional equipment and may not be in the siderail as shown.

- 10. Pull the cables through the siderail (toward the center of the bed).
- 11. Unplug the cable assembly (F) underneath the head section.
- 12. Reverse the above steps to install the new cable.

CAUTION

Be sure to position the cables on both sides of the pivot arm, as shown in the illustration on page 81, before reattaching the pivot arm cover. If not done properly, the cover will not fit tightly and damage could occur to the cables.

SIDERAIL ASSEMBLY REMOVAL

Required Tools:

#2 Phillips Screwdriver

NOTE

Because of fastener changes, it will be necessary to use either a Phillips screwdriver or a T27 Torx to remove the bolts securing the litter access cover.

Removal Procedure:

1. Run the head section up electrically when removing a head end siderail.


T27 Torx

- 2. Unplug the bed power cord from the wall socket.
- 3. Lower the siderail being removed.

NOTE

If removing a head end siderail, the cable must be unplugged from the CPU board under the head end litter access cover.

4. Using a #2 Phillips screwdriver or a T27 Torx, remove the four screws holding the head end litter access cover to the litter and remove the cover.

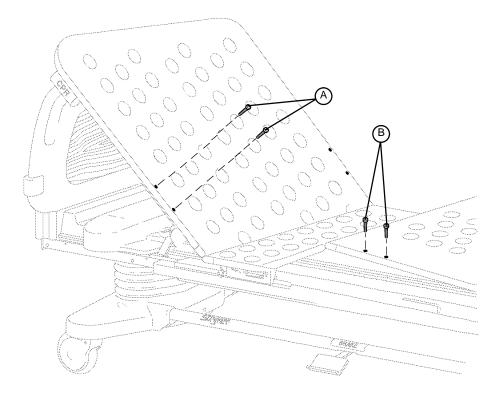

- 5. Using a T27 Torx, remove the two screws (A) or (B) holding the complete siderail assembly on the head or foot section.
- 6. Push in on the siderail assembly to clear under the head or foot section and pull the siderail back to remove it from the bed.
- 7. Reverse the above steps to reinstall the siderail assembly.

NOTE

Grease the glide rods on the replacement siderail assembly with Syntech grease.


FOOT BOARD HINGE REMOVAL

Required Tools:

#2 Phillips Screwdriver

Procedure:

- 1. Using a #2 Phillips screwdriver, remove the screws (A) holding the door and hinge assembly to the foot board.
- 2. If replacing the hinge only, use a Phillips screwdriver to remove the screws holding the hinge to the door.
- 3. Reverse the above steps to attach the replacement door and/or hinge.

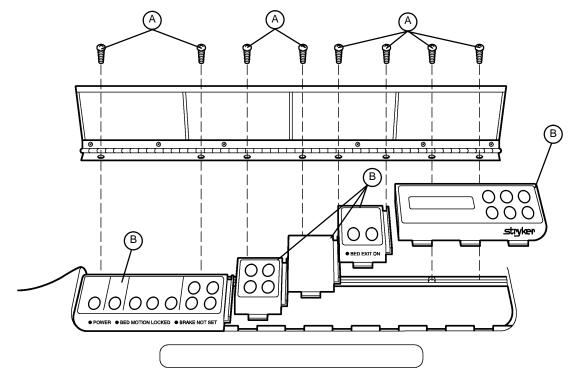

FOOT BOARD MODULE REPLACEMENT

Required Tools:

#2 Phillips Screwdriver

Procedure:

1. Unplug the bed power cord from the wall socket. Remove the foot board hinge (see above).

NOTE

Regardless of which module is being replaced, the farthest module to the right must be removed first.

- 2. Pull the module out of the foot board and disconnect the cable from the module (B).
- 3. Reverse the above steps to install the new module.


CAUTION

The modules must be overlapped as shown in the illustration or fluids could enter the board cavity and cause damage.

FOOT BOARD INTERFACE PLUG REPLACEMENT

Required Tools:

#2 Phillips Screwdriver

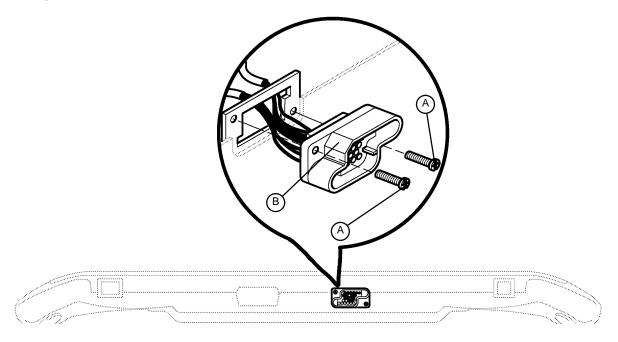


BOTTOM VIEW OF FOOT BOARD


Procedure:


- 1. Unplug the bed power cord from the wall socket.
- 2. Remove the foot board from the bed to access the bottom of the board.
- 3. Properly ground yourself (see page 29 for static discharge precautions).
- 4. Using a #2 Phillips screwdriver, remove the eight screws holding the foot board door to the foot board and remove the door.
- 5. Using a #2 Phillips screwdriver, remove the two screws (A) holding the plug to the foot board.
- 6. Disconnect the cable from the foot board module cable. Note proper placement of the cable so it will be reconnected properly.
- 7. Reverse the above steps to install the new interface plug.


CAUTION


Be sure to install the plug with the flat edge (B) at the top left, as shown in the illustration, or the foot board interface plug will not mate properly with the bed and damage to the plug or foot board could result.

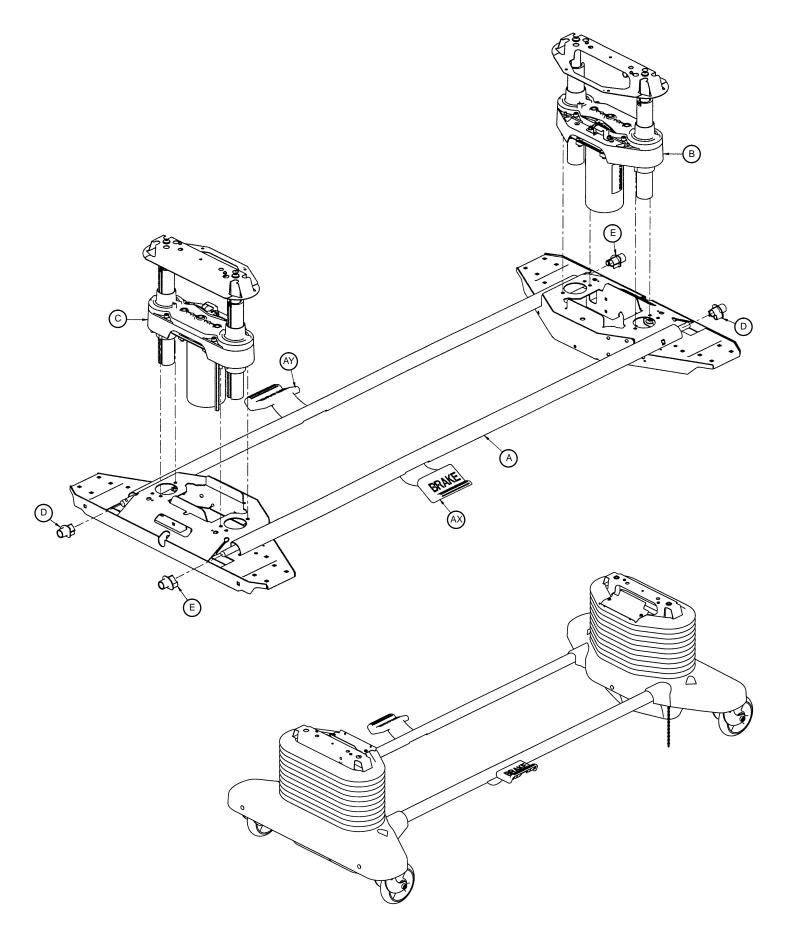


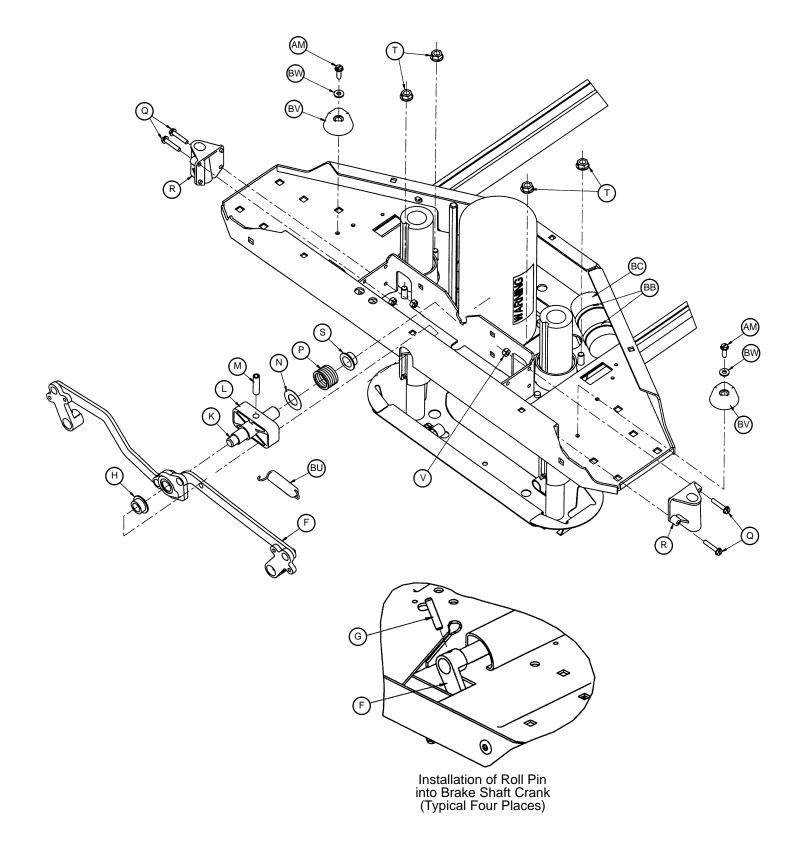

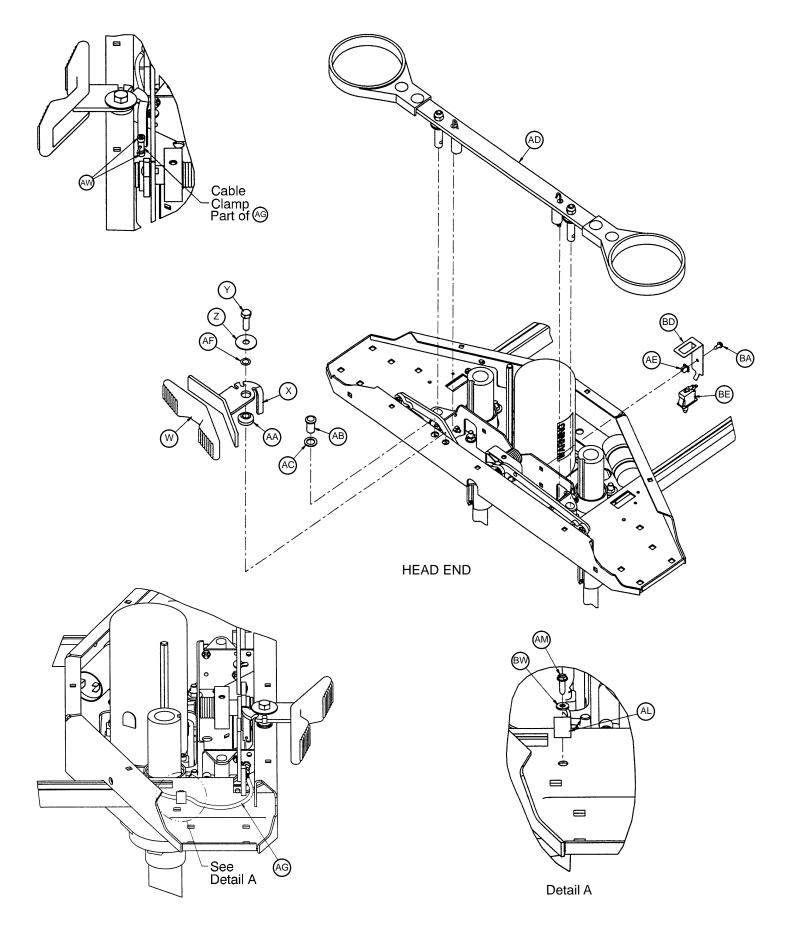


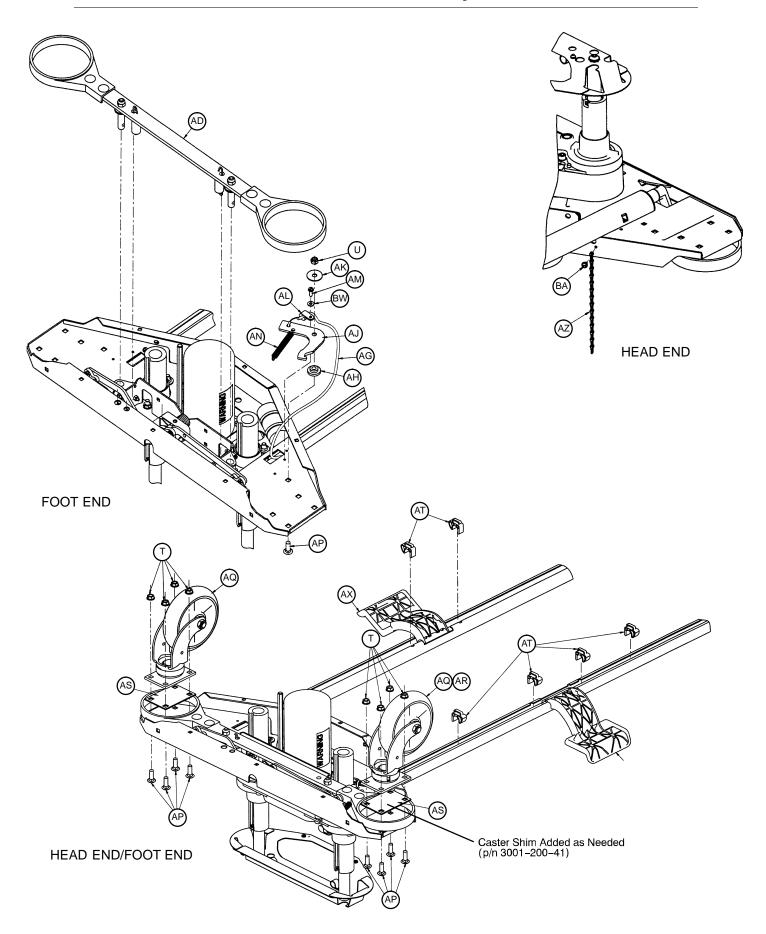

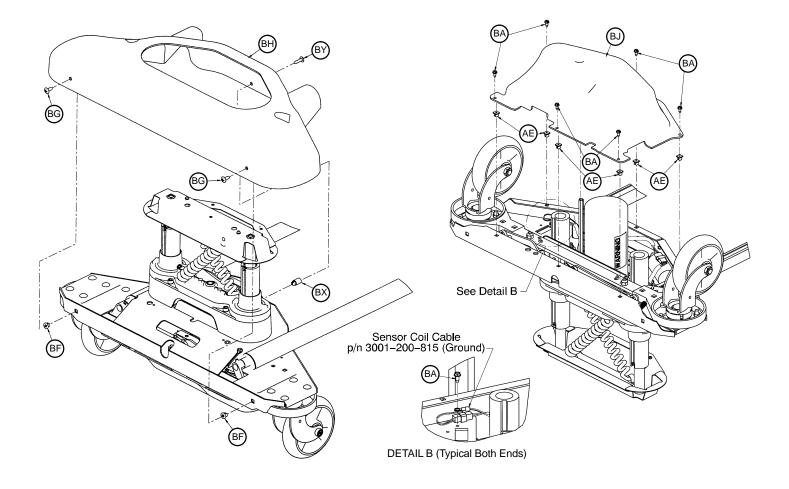


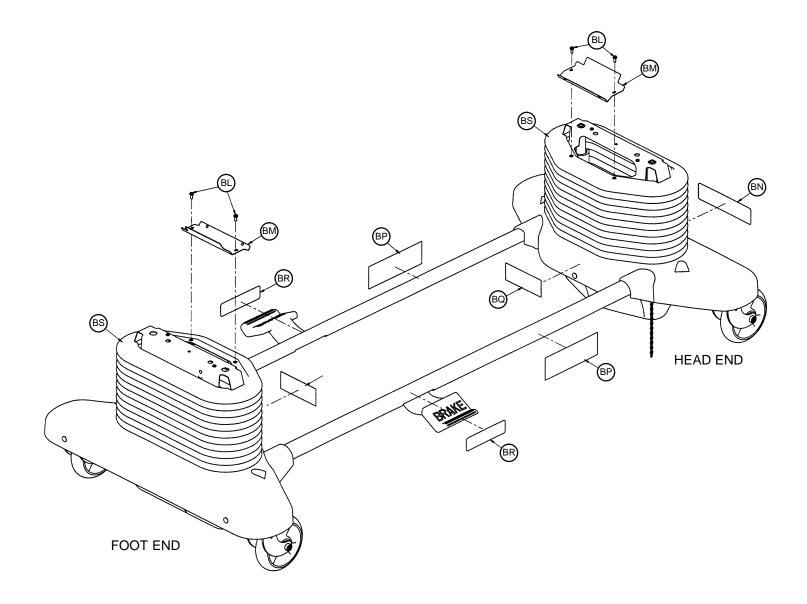



ltem	Part No.	Part Name	Qty.	Item	Part No.	Part Name	Qty.
А	3001-200-101	Base Weldment	1	AN	3000-200-346	Steer Lock Spring	1
В	(page 96)	Lift Ass'y, Head End	1	AP	5–20	Carriage Bolt	17
С	(page 96)	Lift Ass'y, Foot End	1	AQ	3000-200-30	6" Caster	3
D	3000-200-305	Brake Shaft Bushing, Rt.	2		3000-200-36	8" Caster (Option)	3
Е	3000-200-331	Brake Shaft Bushing, Lt.	2	AR	3000-200-16	6" Steer Caster	1
F	(page 98)	Brake Crank Assembly	2		3000-200-37	8" Steer Caster	1
G	26–14	Roll Pin	4	AS	3000-200-41	Caster Shim	4
Н	3001-200-317	Brake Cam Shaft Bushing	2	AT	3001-200-306	Brake Shaft Pedal Brg.	6
K	3001-200-323	Brake Cam Shaft	2	AW	4–298	Soc. Hd. Cap Screw	2
L	3001-200-304	Brake Cam	2	AX	(page 101)	Brake Shaft, Left	1
Μ	3000-200-313	Slotted Roll Pin	2	AY	(page 101)	Brake Shaft, Right	1
Ν	3000-200-311	Cam Shaft Thrust Washer	• 4	AZ	715-1-156	6" Ground Chain	1
Р	52–812	Cam Shaft Comp. Spring	2		3001-200-53	8" Ground Chain	1
Q	3–122	Hex Washer Hd. Screw	8	BA	23–25	Hex Washer Hd. Screw	16
R	3000-200-328	Brake Guide Bushing	4	BB	38–151	Cable Tie	4
S	3001-200-321	Brake Cam Shaft Bushing	2	BC	3000-200-243	Lift Motor Capacitor	2
Т	16–98	Hex Flange Nut	24	BD	3000–200–343	Brake Switch Bracket	1
U	16–11	Flexlock Nut	1	BE	3000-300-58	Plunger Switch	1
V	16–2	Nylock Nut	8	BF	18–36	Plastic Clip Nut	4
W	3000-200-336	Steer Pedal	1	BG	23–92	Truss Hd. Phillips Screw	4
Х	3000-200-335	Steer Pedal Arm	1	BH	3000-200-9	Uni–Pan Cover	2
Y	3–120	Hex Hd. Screw	1	BJ	(page 102)	Bottom Cover Ass'y	2
Z	3000-200-348	Wide Waaher	1	BL	3–224	Hex Washer Hd. Screw	4
AA	3000-200-341	Steer Pedal Bushing	1	BM	3001–200–9	Bellows Bracket	2
AB	3000–200–337	Ball Plunger	1	BN	3000–200–603	Steer Pedal Label	1
AC	3000-200-349	Narrow Washer	1	BP	3000-200-602	Stryker Logo Label	2
AD	(page 99)	Brake Bar Assembly	2	BQ	988–2–708	Caution Label	2
AE	3000-300-2	Plastic Clip Nut	13	BR	3000-200-601	Brake Pedal Label	2
AF	52–305	Flat Washer	1	BS	3000–200–11	Bellows	2
AG	3000-200-342	Steer Cable Ass'y	1	BT	3000–300–113	8" Cable Tie	6
AH	3000–200–339	Steer Lock Lever Bushing	1	BU	3001–200–334	Brake Pedal Ext. Spring	2
AJ	3000-200-340	Steer Lock Lever	1	BV	59–746	Mounting Feet	4
AK	3000-200-347	Special Washer	1	BW	11–302	Flat Washer	6
AL	34–254	Cable Clamp	2	BX	52-800	Mounting Standoff	2
AM	23–101	Hex Washer Hd. Screw	6	BY	23–264	Truss Hd. Screw	2

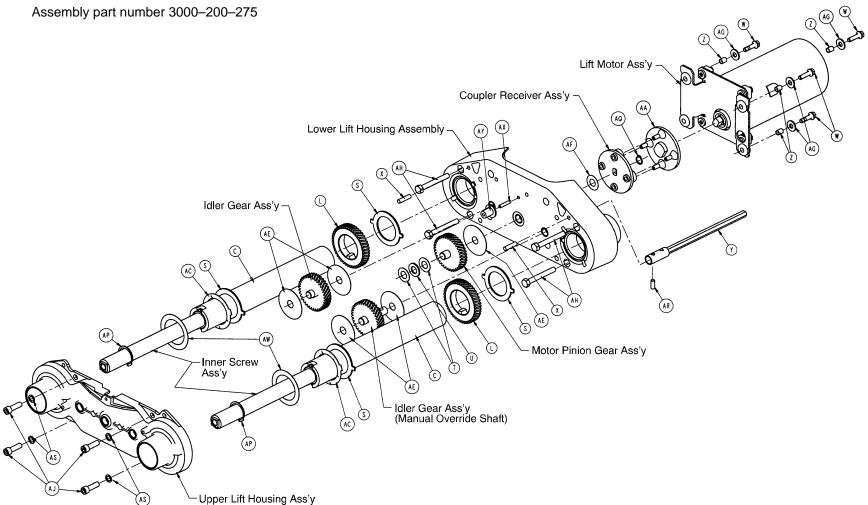


Lift Assembly (Common)

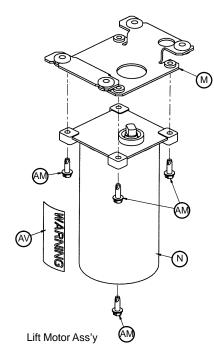

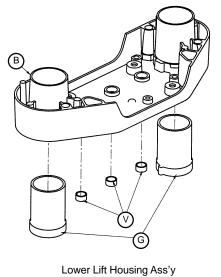


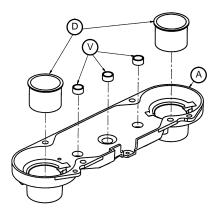


Upper Lift Housing Ass'y

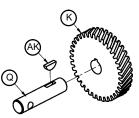


Idler Gear Ass'y (Manual Override Shaft)

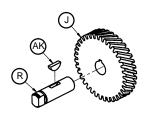


Motor Pinion Gear Ass'y

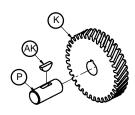

Idler Gear Ass'y

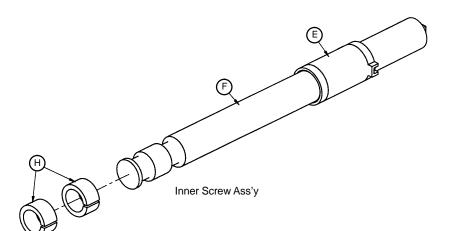

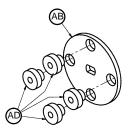


Coupler Receiver Ass'y

Lift Assembly (Common)

ltem	Part No.	Part Name	Qty.
А	3000-200-201	Upper Lift Housing	1
В	3000-200-202	Lower Lift Housing	1
С	3000–200–251	Outer Screw	2
D	3000-200-204	Upper Housing Sleeve	2
E	3000-200-205	Upper Stage Nut	2
F	3000-200-249	Inner Screw	2
G	3000-200-207	Lower Stage Nut	2
Н	3000-200-208	Glide Bushing	4
J	3000-200-209	Motor Pinion Gear	1
K	3000-200-210	Idler Gear	2
L	3000-200-252	Output Gear	2
Μ	(page 97)	Motor Isolation Plate Ass'y	1
Ν	3000-200-213	Lift Motor	1
	3221–200–213	230V Lift Motor	1
Р	3000-200-218	Idler Shaft, Lift	1
Q	3000-200-219	Idler Man. Over. Shaft	1
R	3000-200-220	Input Pinion Shaft	1
S	3000-200-223	Output Gear Thr. Washer	4
Т	3000-200-224	Input Gear Thr. Washer	2
U	3000-200-225	Input Pinion Thr. Bearing	1
V	3000-200-226	Pinion Shaft Bushing	6
W	3001-200-228	Mounting Standoff	4
Х	26–231	Dowel Pin	2
Y	(page 100)	Man. Override Shaft Ass'y	1
Z	3001–300–19	Isolation Sleeve	4
AA	3000-200-233	Lift Motor Coupler	1
AB	3000-200-234	Coupler Receiver	1
AC	3000-200-241	Crush Washer	2
AD	3000-300-455	Isolation Bushing	4
AE	3000-200-245	Gear Washer	5
AF	3000-200-246	Nylon Washer	1
AG	11–408	Flat Washer	4
AH	3–82	Hex Hd. Cap Screw	4
AJ	4–213	Soc. Hd. Cap Screw	4
AK	58–44	Woodruff Key	3
AM	3–331	Hex Washer Hd. Screw	4
AP	28–121	Retaining Ring	2
AQ	28–1	Retaining Ring	1
AR	26–178	Roll Pin	1
AS	11–308	Serrated Belleville Washer	4
AV	3000-300-604	Warning Label	1
AW	3001–200–242	Shim Washer	2
AX	3000–200–239	Pot. Drive Gear Shaft	1
AY	3000–200–216	Potentiometer Drive Gear	1


Lift Assembly (Head & Foot End)

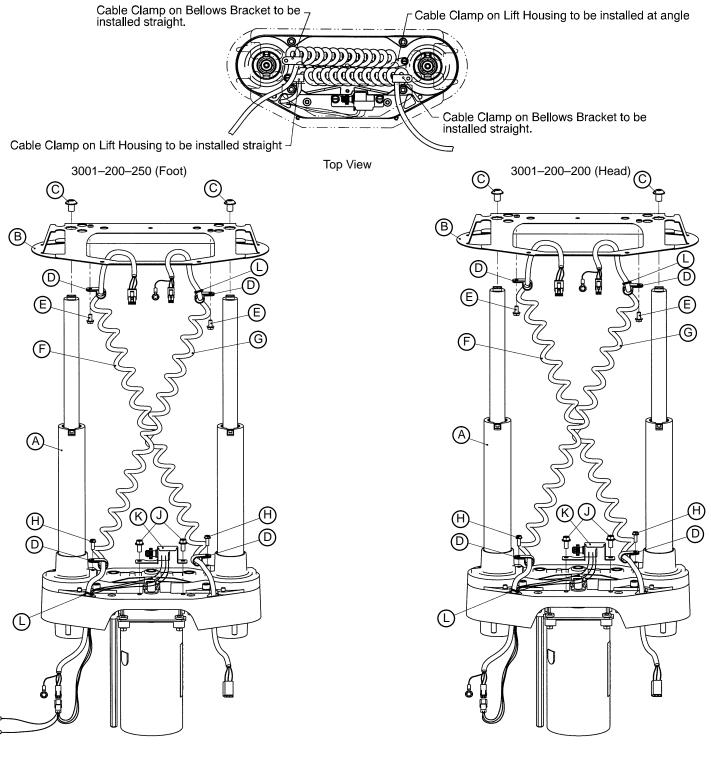

ltem	Part No.	Part Name	Qty.
А	(page 93–95)	Common Lift Assembly	1
В	3000–200–52	Bellows Bracket	1
С	4–245	Flanged But. Hd. Screw	2
D	34–22	Cord Clamp	4
Е	3–123	Hex Washer Hd. Screw	2
F	3001–200–824	Power Coil Cord	1


ltem	Part No.	Part Name	Qty.
G	3001–200–815	Sensor Coil Cord	1
Н	3–128	Hex Washer Hd. Screw	2
J	3–121	Hex Washer Hd. Screw	2
K	(page 97)	Head End Pot. Ass'y	1
K	(page 97)	Foot End Pot. Ass'y	1
L	3000–300–113	Cable Tie	2

Assembly part number 3001–200–214

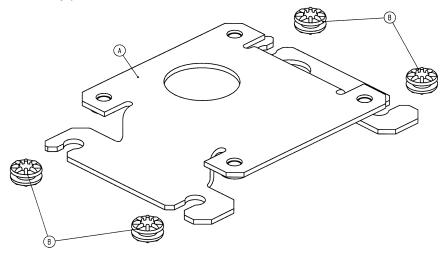


ltem	Part No.	Part Name	Qty.
A	3001–200–213	Isolation Plate	1
B	3000–300–442	Grommet	4


Head and Foot End Potentiometer Assembly

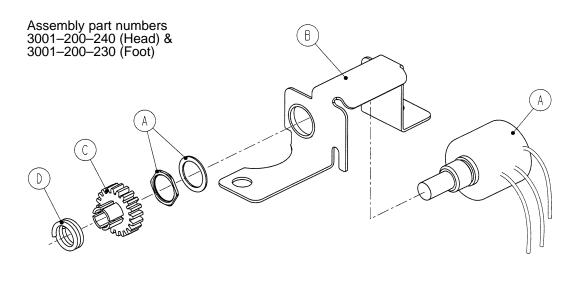


ltem	Part No.	Part Name	Qty.
А	3000-200-807	Head Pot. Cable	1
В	3000-200-217	Pot. Mounting Bracket	1
С	3000-200-215	Pot. Worm Gear	1
D	3000-200-253	Pot. Worm Gear Retainer	1

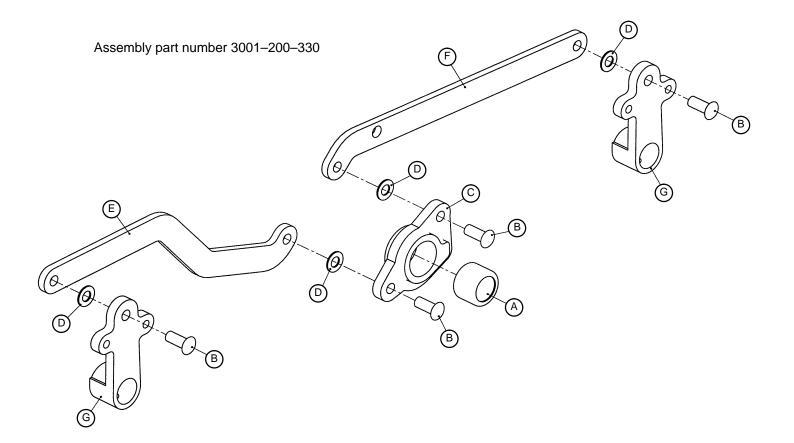


Item	Part No.	Part Name	Qty.
А	3000-200-338	Drawn Cup Roller Clutch	1
В	25–146	Rivet	4
С	3000-200-322	Brake Crank Wheel	1
D	14–4	Nylon Washer	4
Е	3001–200–312	Dog Leg Brake Link	1
F	3001–200–311	Brake Link	1
G	3000-200-302	Brake Shaft Crank	2

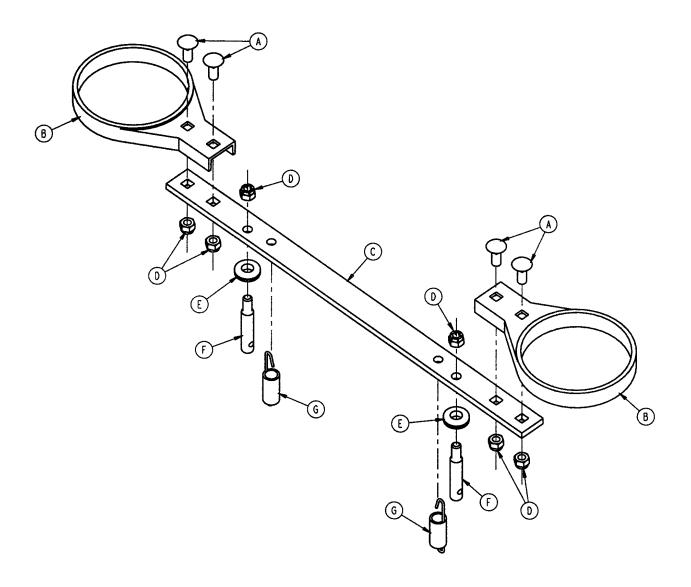

ltem	Part No.	Part Name	Qty.
А	5–18	Carriage Bolt	4
В	3000-200-321	Brake Ring	2
С	3000-200-323	Brake Bar	1
D	16–35	Nylock Nut	6
E	3000-200-324	Brake Bar Bumper	2
F	3000–200–318	Guide Pin	2
G	3000-200-352	Brake Bar Return Spring	2

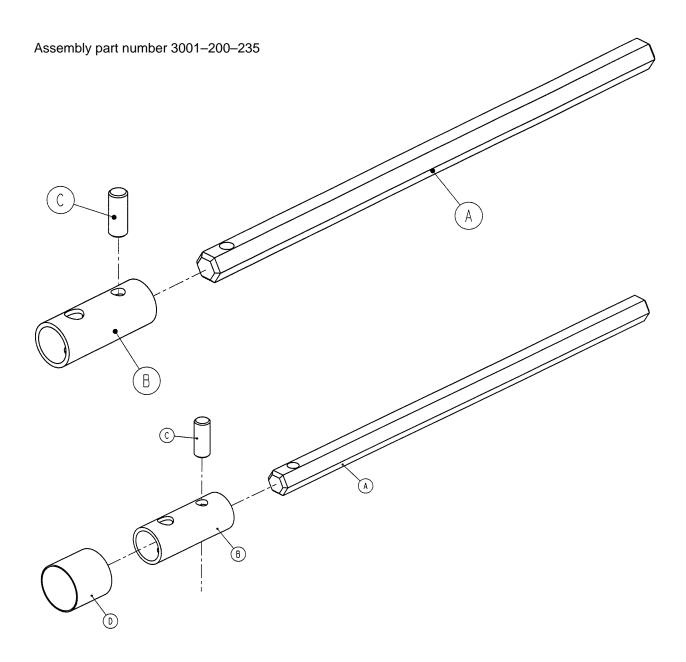


ltem	Part No.	Part Name	Qty.
А	3000-200-235	Manual Override Drive Shaft	1
В	3000-200-236	Manual Override Coupler	1
С	26–172	Roll Pin	1
D	7900–1–322	Heat Shrink Tubing	1

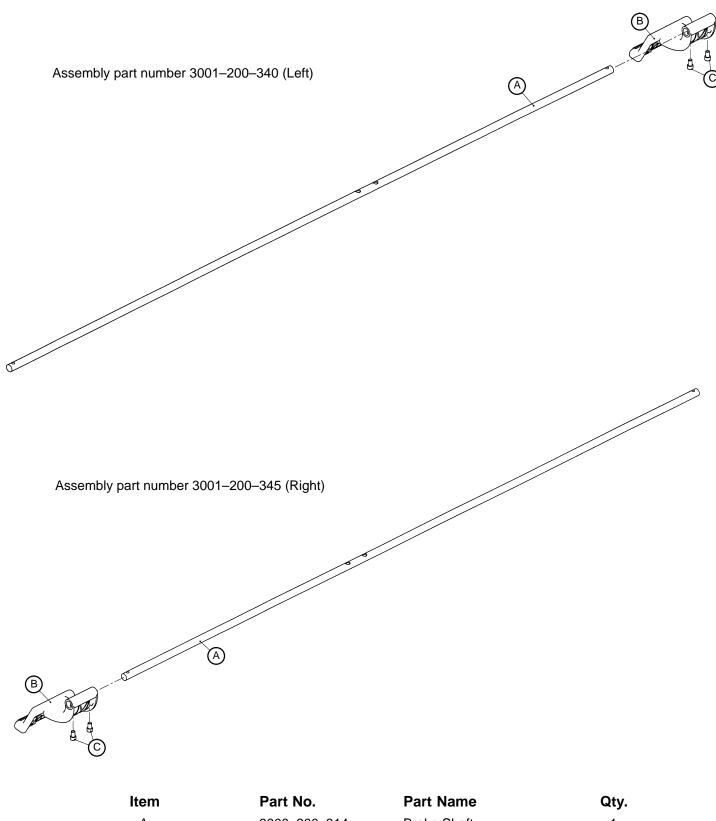



	i art ivo.		œty.
А	3000–200–314	Brake Shaft	1
В	3001–200–325	Brake Pedal	1
С	4–270	Soc. Hd. Cap Screw	2


Assembly part number 3001–200–25

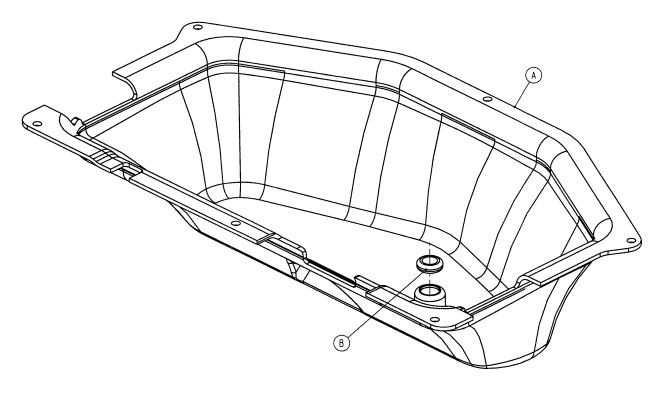



ltem	Part No.	Part Name	Qty.
А	3000-200-1	Bottom Cover	1
В	3000–000–39	Grommet	1

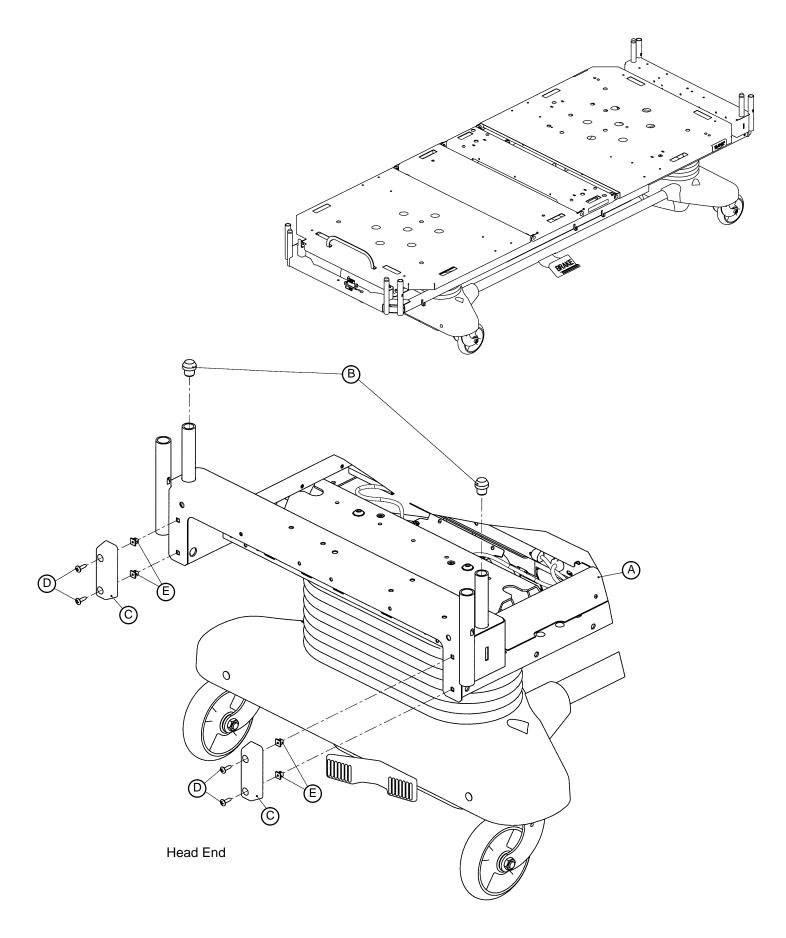


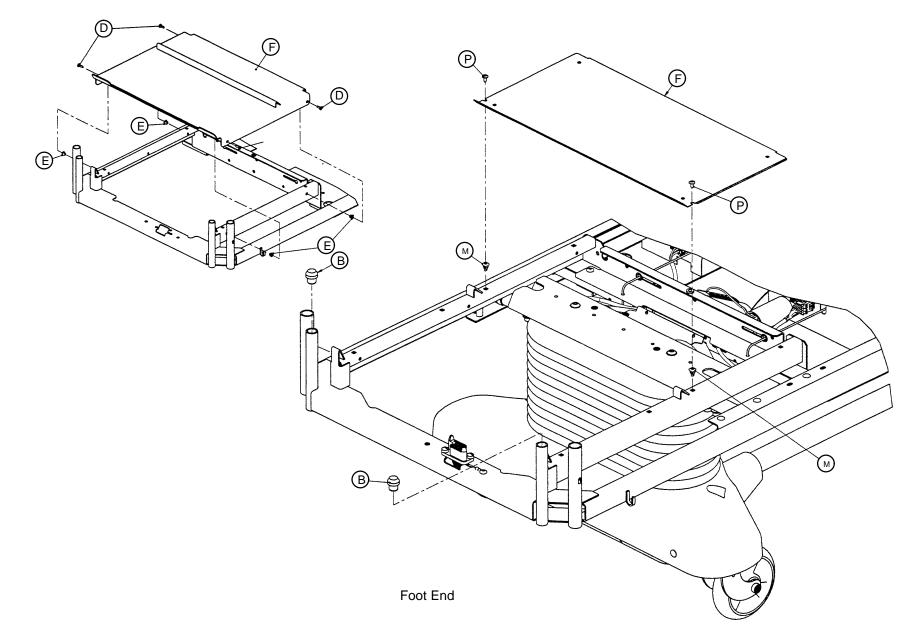


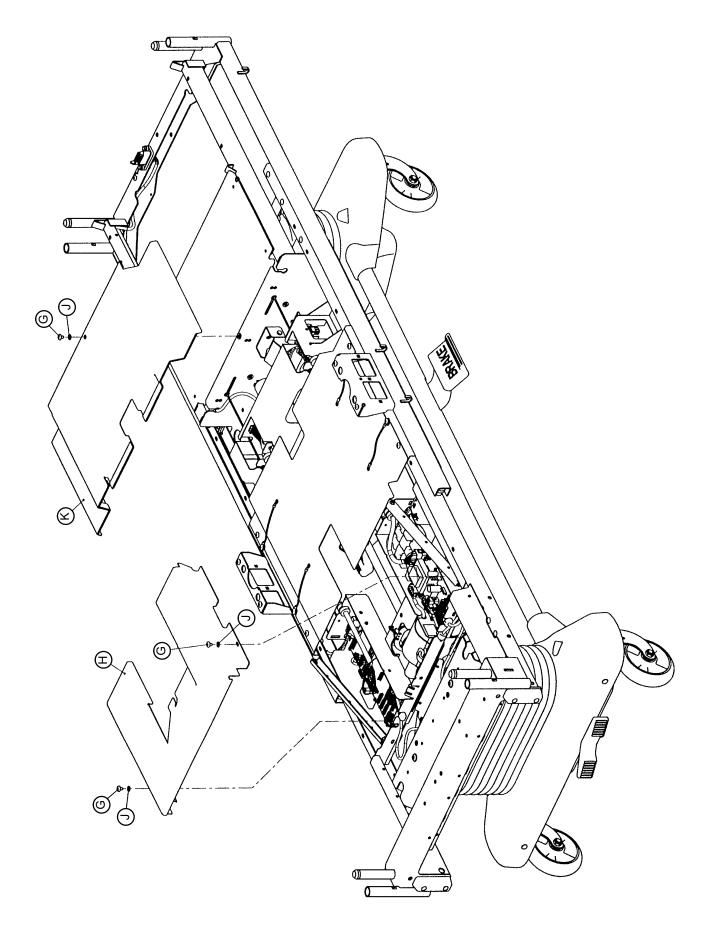


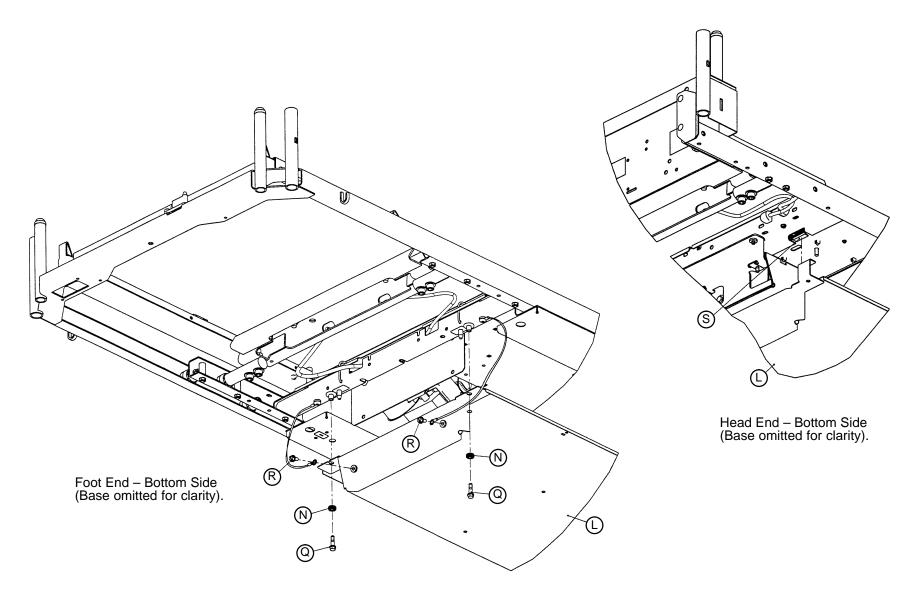

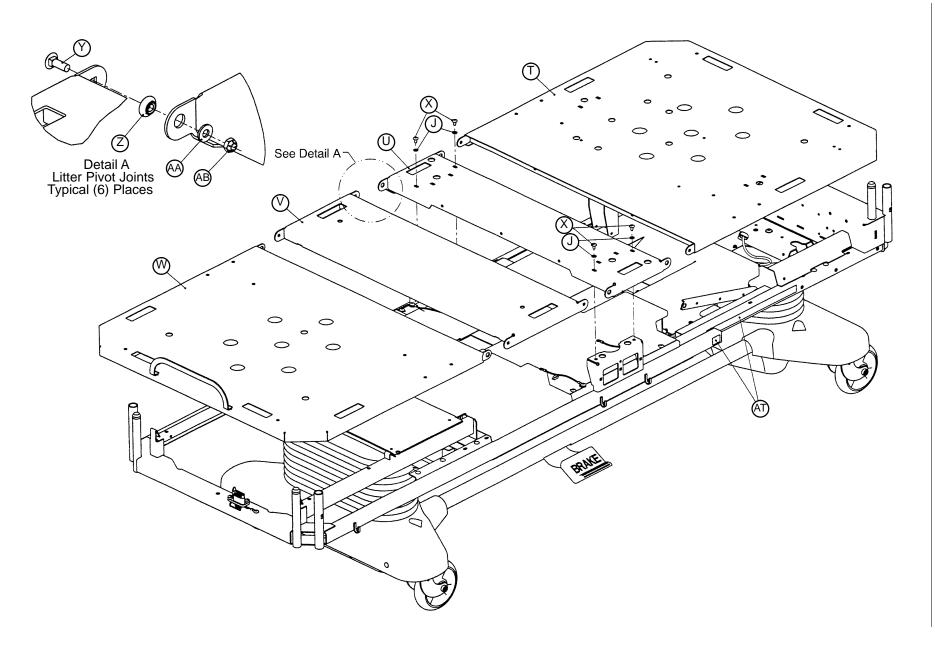

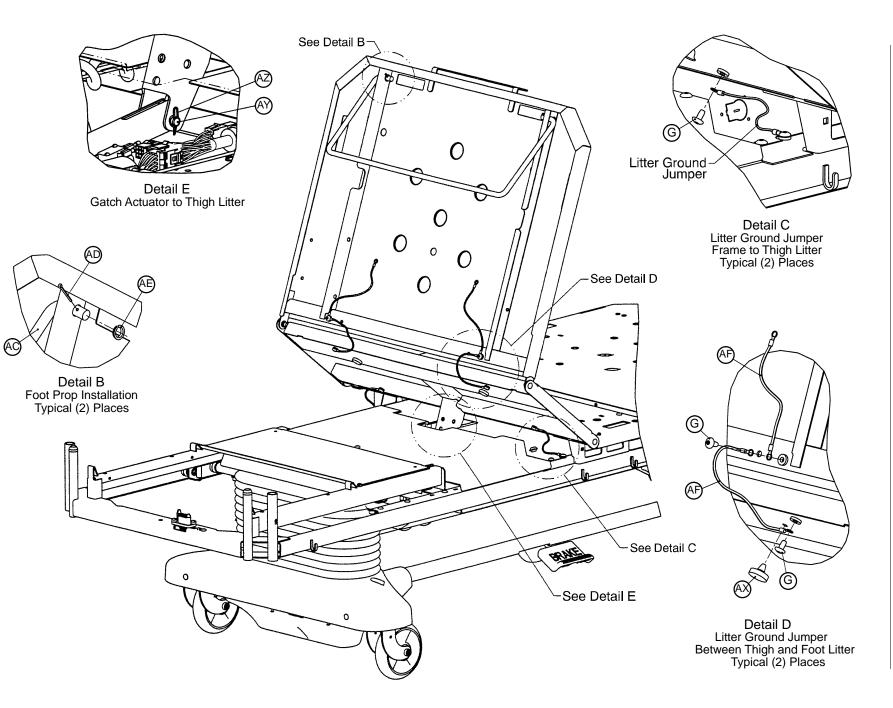

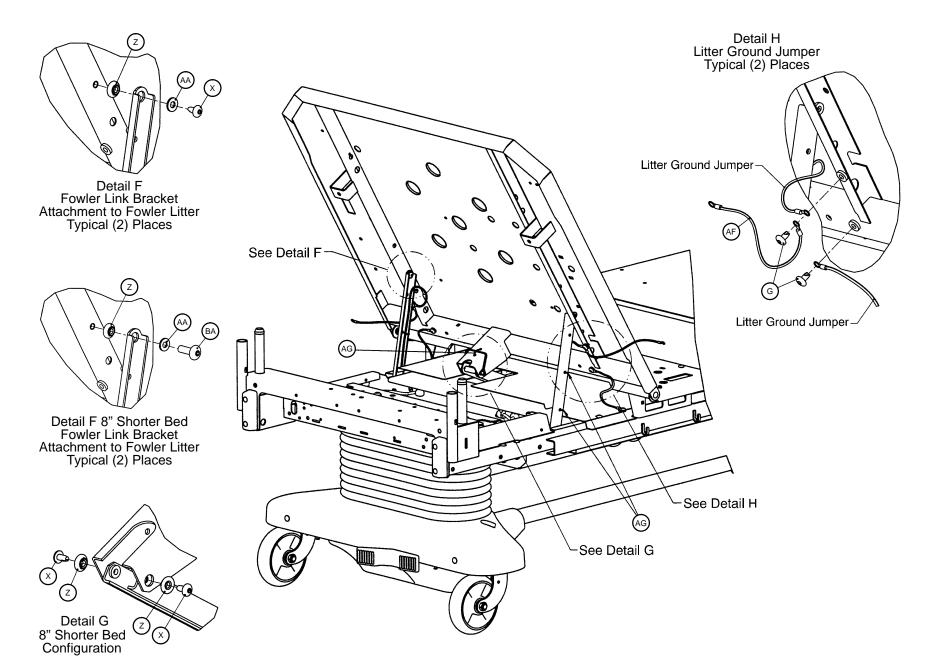

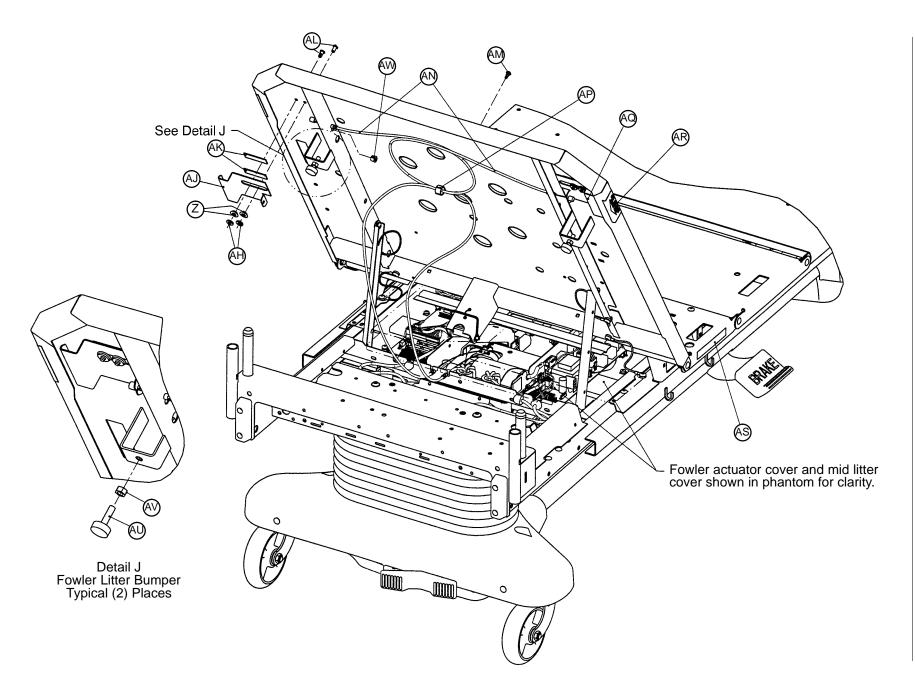

Litter Assembly

ltem	Part No.	Part Name	Qty.	ltem	Part No.	Part Name	Qty.
А	(page 126-132)	Electrical Litter Ass'y	1	AE	52–754	Bushing	2
В	3000–300–349	Сар	4	AF	3001-300-870	Litter Ground Jumper	6
С	3000-300-350	Head End Bumper	2	AG	3000-300-113	Cable Tie	5
G	7–53	Truss Hd. Screw	13	AH	11–376	Washer	4
Н	3001-300-95	Fowler Act. Cover Ass'y	1	AJ	3001-300-662	CPR Handle, Left	1
J	13–10	Ext. Tooth Lock Washer	7	AK	3001–300–663	Velcro Strip	4
K	(page 112)	Electrical Cover Ass'y	1	AL	25–120	Pop Rivet	4
L	3001-300-75	Motion Interrupt Ass'y	1	AM	3000–300–477	CPR Conduit Stud	1
Ν	3000-200-228	Motor Mount Grommet	2	AN	3001–300–670	CPR Cable Ass'y	2
Q	3001–200–228	Mounting Standoff	2	AP	59–743	Wire Harness Clip	1
R	3–221	Hex Washer Hd. Screw	2	AQ	3001–300–661	CPR Handle, Right	1
S	3001-300-14	Edge Grommet	2	AR	3001-300-603	CPR Release Label	2
U	3001–300–140	Litter Seat	1	AS	2020-88-820	Caution Label	2
V	3001-300-165	Thigh Litter Ass'y	1	AU	3000–300–3	Bumper	2
Х	7–52	Truss Hd. Screw	6	AV	15–12	Hex Nut	2
Y	5–17	Carriage Bolt	6	AW	30–52	Snap Bushing	2
Z	3001–300–99	Fowler Bushing	12	AX	3001–300–8	Thigh Bumper	2
AA	11–303	Flat Washer	8	AY	26–277	Clevis Pin	1
AB	16–16	Fiber Lock Nut	6	AZ	27–22	Rue Ring	1
AD	27–7	Cotter Pin	2				


Litter Assembly – Standard Components

Litter Assembly – 8" Shorter Bed

Litter Assembly – Standard Bed


Item	Part No.	Part Name	Qty.	ltem	Part No.	Part Name	Qty.
А	(page 126–132)	Electrical Litter Ass'y	1	А	(page 126-132)	Electrical Litter Ass'y	1
D	23–80	Truss Hd. Screw	8	D	23–80	Truss Hd. Screw	8
Е	3000-300-2	Plastic Clip Nut	8	Е	3000-300-2	Plastic Clip Nut	8
F	(page 114)	Foot Cover Assembly	1	F	(page 113)	Foot Cover Ass'y	1
Т	3001-342-115	Fowler Litter Ass'y	1	Μ	52-816	Screw Grommet	2
W	3001-342-180	Foot Litter Assembly	1	Р	23–265	Truss Hd. Screw	2
Z	3001-300-99	Fowler Bushing	2	Т	3001-300-105	Fowler Litter Ass'y	1
AC	3001-342-162	Foot Prop Rod	1	W	3001-300-180	Foot Litter Ass'y	1
AT	3001-342-603	Fowler Angle Label	1	AC	3001-300-162	Foot Prop Rod	1
BA	7–56	Truss Hd. Screw	2	AT	3000-300-602	Fowler Angle Label	1

Assembly part number 3001-300-30

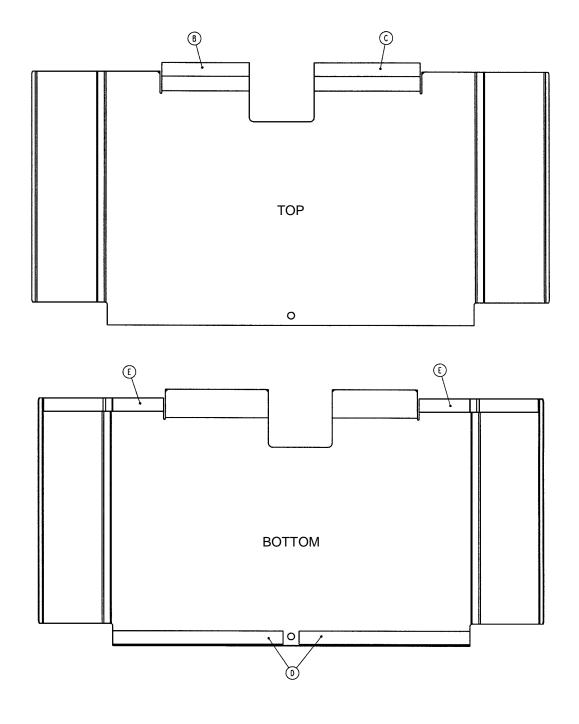


Item	Part No.	Part Name	Qty.
А	3001-300-32	Electrical Cover	1
В	7900–1–291	4.75" Foam Tape	1
С	7900–1–291	5.75" Foam Tape	1
D	7900–1–291	9.5" Foam Tape	2
E	7900–1–291	7.5" Foam Tape	2

Assembly part number 3001–300–46

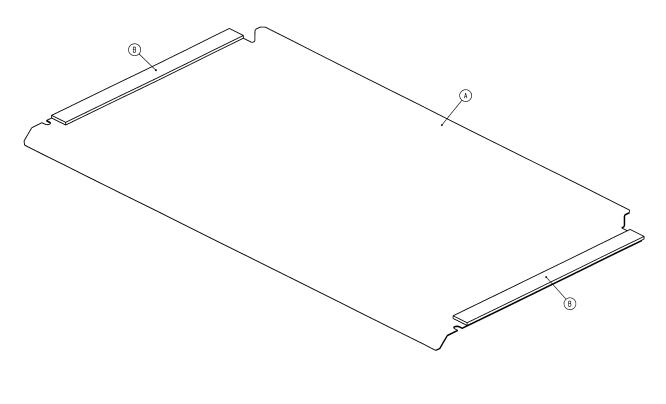



ltem	Part No.	Part Name	Qty.
А	3001–300–47	Foot Cover	1
В	7900–1–291	9.5" Foam Tape	2

Assembly part number 3001–342–45

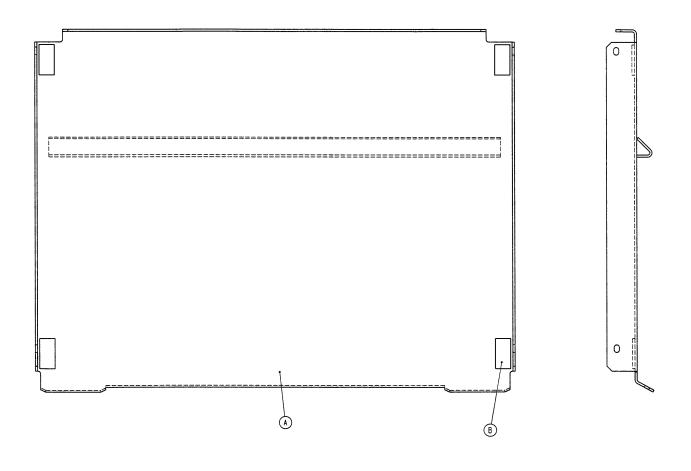



ltem	Part No.	Part Name	Qty.
A B	3001–342–46 7900–1–291	Foot Cover .75" Foam Tape	1 4

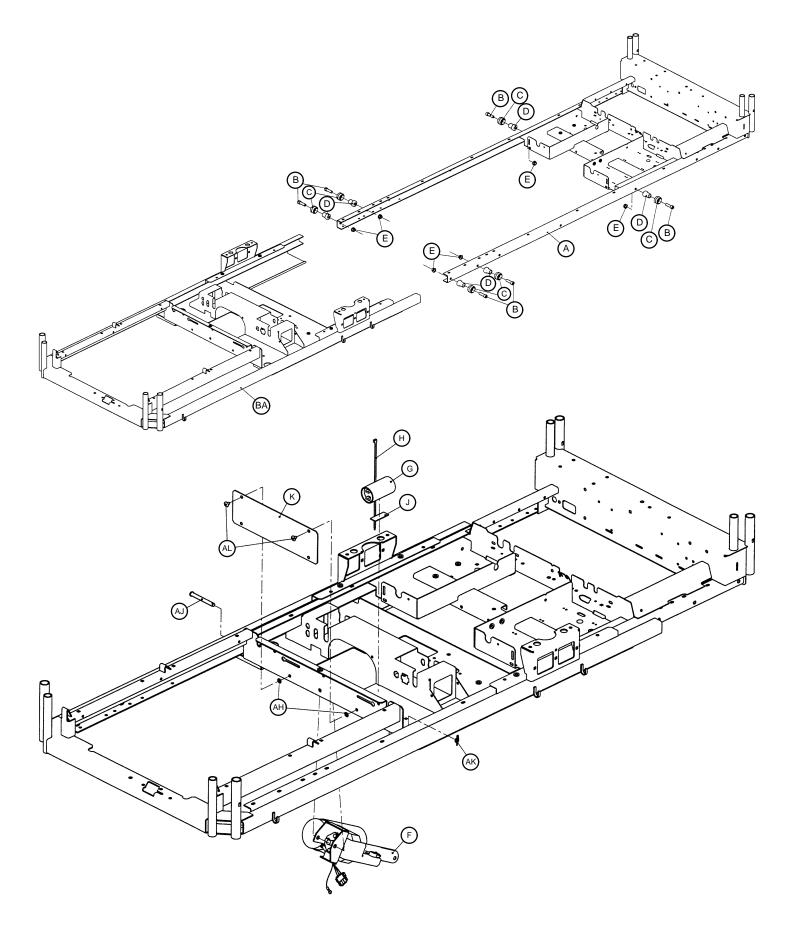

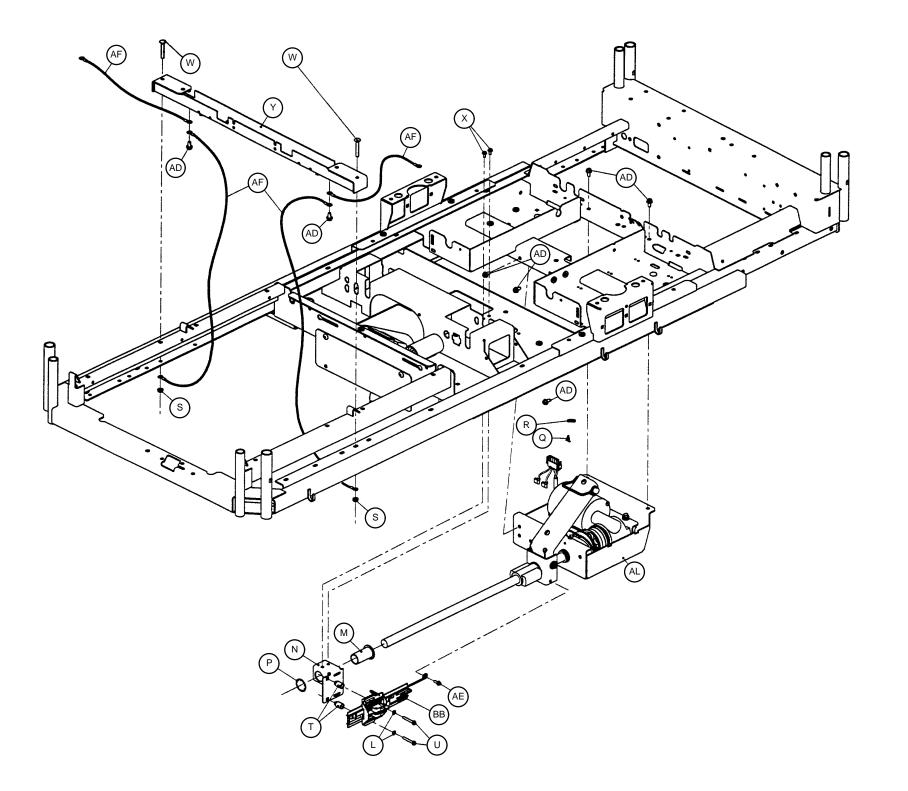

 \cap

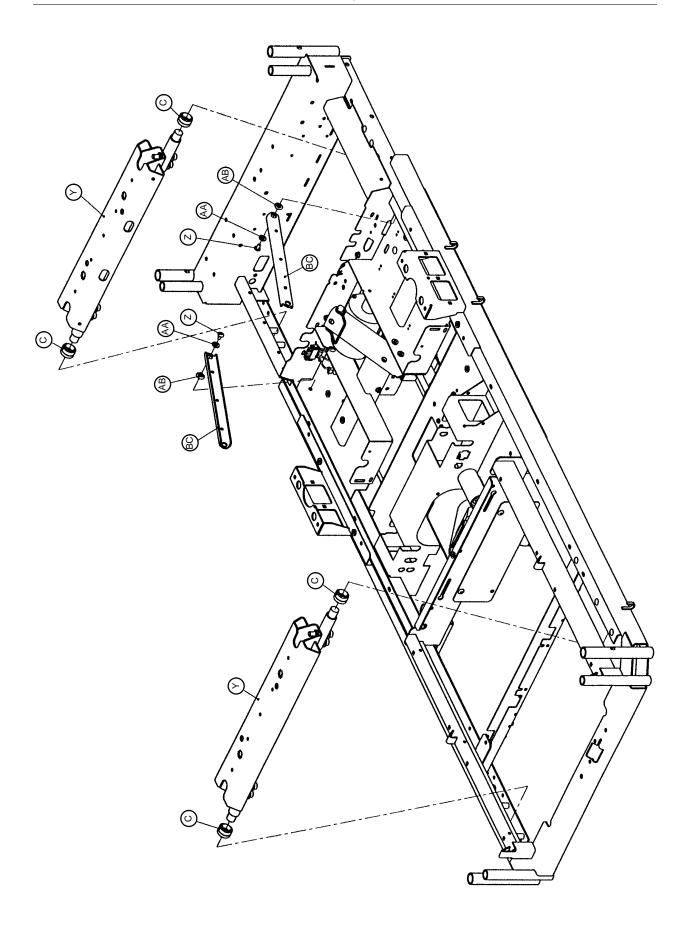

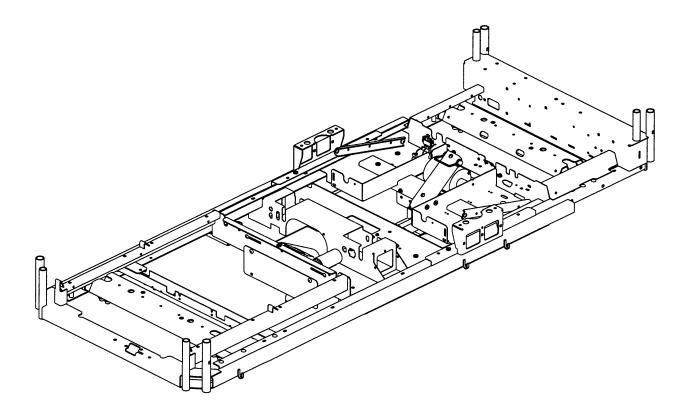


Mechanical Litter Assembly – Standard Components

Item	Part No.	Part Name	Qty.	Item	Part No.	Part Name	Qty.
А	3001-300-302	Stationary Frame Ass'y	1	U	23–251	Hex Washer Hd. Screw	2
В	4–211	Soc. Hd. Cap Screw	6	V	3001-300-335	Cross Support Ass'y	1
С	3000-300-353	Roller	10	W	5–22	Carriage Bolt	10
D	3000-300-352	Roller Shaft	6	Х	3–123	Hex Washer Hd. Screw	2
Е	16–93	Flange Lock Nut	6	Y	(page 120 & 121)	Std. Lift Header Ass'y	2
F	(page 119)	Gatch Motor Ass'y	1	Z	7–52	Truss Hd. Torx	2
G	3000-300-401	Capacitor	1	AA	11–303	Flat Washer	2
Н	38–151	11" Cable Tie	1	AB	3001–300–99	Fowler Bushing	2
J	44–29	Double-Sided Tape	1	AC	3001–300–4	Stationary Frame Space	8
K	3001–300–418	Gatch Shield	1	AD	3–221	Hex Washer Hd. Screw	7
L	13–18	Ext. Tooth Lock Washer	2	AE	23–82	Hex Washer Hd. Screw	1
Μ	3000–300–9	Screw Support Bushing	1	AF	3001-300-870	Litter Ground Jumper	4
Ν	3001–300–498	Screw Support Bracket	1	AG	(page 122 & 123)	Fowler Drive Ass'y	1
Р	28–122	Retaining RIng	1	AH	13–10	Ext. Tooth Lock Washer	2
Q	52–764	Clip	1	AJ	26–279	Clevis Pin	1
R	3001–300–664	CPR Spring	1	AL	27–22	Rue Cotter Pin	1
S	16–6	Kep Nut	10	AM	7–53	Torx Screw	2
Т	52–800	Grommet	2				


Litter Assembly – 8" Shorter Bed

Litter Assembly – Standard Bed

ltem	Part No.	Part Name	Qty.	ltem	Part No.	Part Name	Qty.
AL	3001-342-455	Fowler Drive Ass'y	1	AL	3001–343–455	Fowler Drive Ass'y	1
BA	3001-342-203	Moving Frame Assembly	1	BA	3001-300-203	Moving Frame Assembly	/ 1
BB	(page 125)	Fowler Limit Assembly	1	BB	(page 124)	Fowler Limit Ass'y	1
BC	3000-342-7	Fowler Link Assembly	2	BC	3000-300-6	Fowler Link Assembly	2

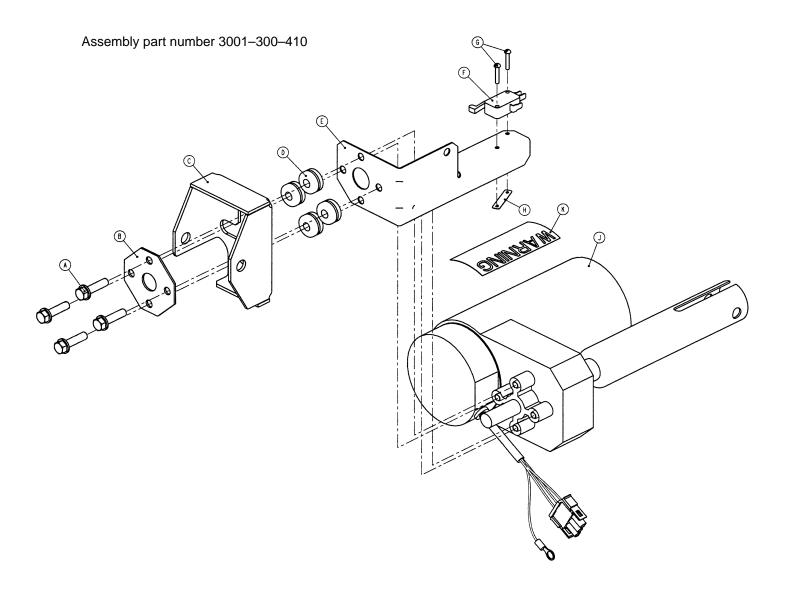

Item	Part No.	Part Name	Qty.
А	3001-200-228	Mounting Standoff	4
В	3001–300–499	Gatch Motor Mounting Plate	1
С	3001-300-235	Motor Mount Bracket	1
D	52–817	Grommet	4
E	3001–300–421	Gatch Limit Bracket	1
F	3000-300-41	Micro Switch	1
G	2–51	Round Hd. Mach. Screw	2
Н	16–69	Twin–Type Fastener	1
J	2025–232–70	Gatch Motor	1
	2026-232-70	230V Gatch Motor	1
К	3000-300-604	Warning Label	1

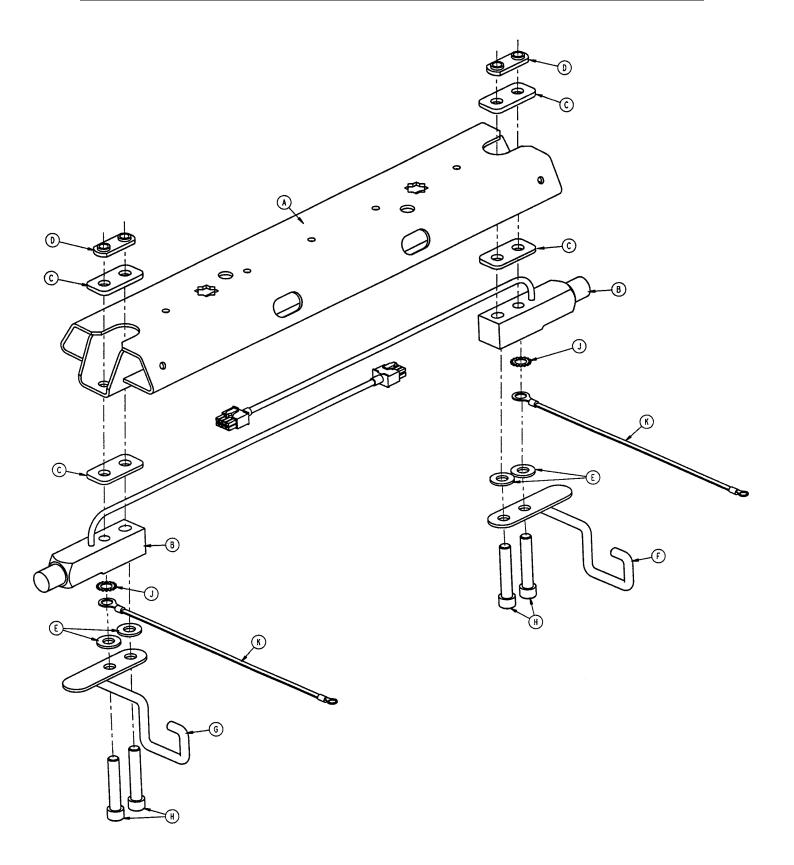

Lift Header Assembly – Standard Bed

ltem	Part No.	Part Name	Qty.
А	3001–300–525	Lift Header Assembly	1
В	3001–300–511	"Imitation" Load Cell	2
D	3001-300-509	Lift Header Weld Nut	2
E	11–301	Flat Washer	4
Н	4–288	Hex Soc. Hd. Cap Screw	4

Lift Header Assembly – Scale Bed


ltem	Part No.	Part Name	Qty.
А	3001–300–525	Lift Header Assembly	1
В	3001-307-55	Load Cell	2
С	3001-307-21	Load Cell Shim	4
D	3001-300-509	Lift Header Weld Nut	2
E	11–301	Flat Washer	4
Н	4–288	Hex Soc. Hd. Cap Screw	4
J	13–32	External Tooth Lock Washer	2
К	3001–300–871	Ground Jumper	2

Lift Header Assembly – Scale Bed/Isolated Foley Bag Hooks

Item	Part No.	Part Name	Qty.
А	3001-300-525	Lift Header Assembly	1
В	3001–307–55	Load Cell	2
D	3001-300-509	Lift Header Weld Nut	2
E	11–301	Flat Washer	2
F	3001-326-20	Isolated Foley Bag Ass'y, Rt.	1
G	3001-326-10	Isolated Foley Bag Ass'y, Lt.	1
Н	4–288	Hex Soc. Hd. Cap Screw	4
J	13–32	External Tooth Lock Washer	2
К	3001–300–871	Ground Jumper	2

Lift Header Assembly – Standard Bed/Isolated Foley Bag Hooks


ltem	Part No.	Part Name	Qty.
А	3001–300–525	Lift Header Assembly	1
В	3001–300–511	"Imitation" Load Cell	2
D	3001-300-509	Lift Header Weld Nut	2
E	11–301	Flat Washer	4
F	3001-326-20	Isolated Foley Bag Ass'y, Rt.	1
G	3001-326-10	Isolated Foley Bag Ass'y, Lt.	1
Н	4–288	Hex Soc. Hd. Cap Screw	4

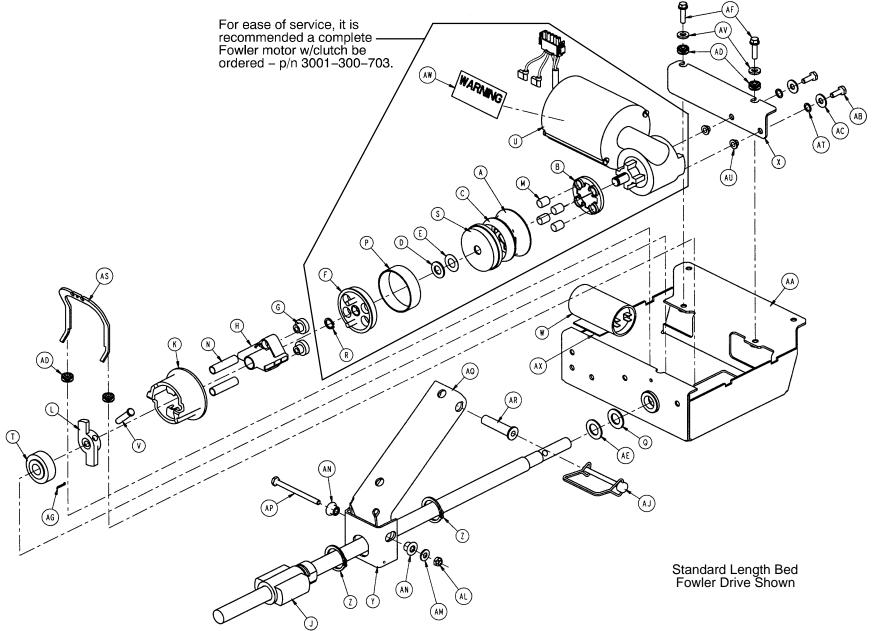

Fowler Drive Assembly

Fowler Drive Assembly

ltem	Part No.	Part Name	Qty.	ltem	Part No.	Part Name	Qty.
А	3000-300-451	CPR Spring Cup	1	W	3000-300-453	Motor Capacitor	1
В	3000-300-470	CPR Spring Seat	1		3221-300-401	230V Motor Capacitor	1
С	3000-300-452	CPR Brake Disc	1	Х	3001-300-466	Actuator Mounting Brkt.	1
D	3000-200-224	Idler Gear Thrust Washer	1	Z	28–120	External Retaining Ring	2
Е	3000-200-225	CPR Thrust Bearing	1	AA	3001–300–475	Fowler Actuator Wldmt.t	1
F	3001–300–454	CPR Coupler	1	AB	3–214	Hex Hd. Cap Screw	2
G	3000-300-455	CPR Isolation Bushing	2	AC	11–3	Flat Washer	2
Н	3000-300-456	CPR Isolator	1	AD	3000-300-442	Fowler Drive Grommet	4
J	3000-300-457	CPR Ball Screw	1	AE	3000–300–482	Thrust Washer	1
K	3000-300-461	CPR Decoupler	1	AF	3001–200–228	Mounting Standoff	2
L	3000-300-462	CPR Wing	1	AG	27–17	Cotter Pin	1
Μ	3000–300–463	CPR Brake Spring	4	AK	3000–300–114	Cable Tie	2
Ν	3000-300-464	CPR Engagement Spring	2	AL	16–16	Fiberlock Nut	1
Р	3000-300-465	CPR Clutch Spring	1	AM	11–303	Flat Washer	1
Q	3000-300-466	Thrust Bearing	1	AN	3000–300–1	Litter Pivot Bushing	2
R	28–131	External Retaining Ring	1	AP	3–206	Hex Hd. Cap Screw	1
S	3000-300-469	Brake Cup	1	AS	3000–300–658	CPR Release Arm	1
Т	3000-300-471	Roller Bearing	1	AT	52–719	Nylon Flat Washer	2
U	3001–300–472	CPR Motor	1	AU	52–718	Nylon Insulator	2
	3221–300–472	230V CPR Motor	1	AV	11–304	Shim Washer	2
V	3000-300-473	Clevis Pin	1	AW	3000–300–604	Warning Label	1
				AX	44–29	Double–Stick Tape	1

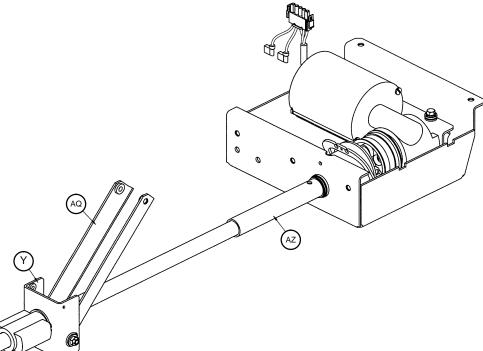


8" Shorter Bed Fowler Drive Shown

Fowler Actuator – Standard Bed


Fowler Actuator – 8" Shorter Bed

ltem	Part No.	Part Name	Qty.	ltem	Part No.	Part Name	Qty.
Y	3001-342-498	Fowler Link Bracket	1	Y	3001-300-497	Fowler Link Bracket	1
AQ	3001-342-495	Fowler Link	1	AJ	3000-300-424	Hitch Pin	1
AZ	3001-342-458	Fowler Nut Stop Tube	1	AQ	3001-300-495	Fowler Link	1
				AR	3000-300-420	Link Bushing	1

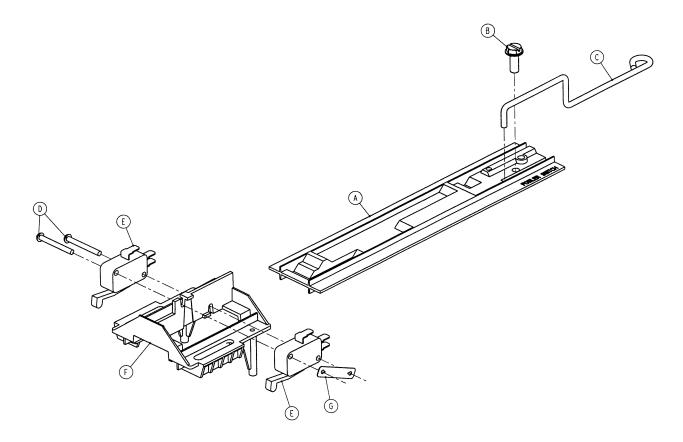



ltem	Part No.	Part Name	Qty.
А	3000-300-36	Fowler Cam Card	1
В	23–82	Hex Washer Hd. Screw	1
С	3001-300-37	Link Wire	1
D	2–3	Round Hd. Screw	2
E	3000-300-41	Micro Switch	2
F	3000-300-44	Cam Guide	1
G	16–69	Twin Fastener	1


8" Shorter Bed Fowler Limit Switch Assembly

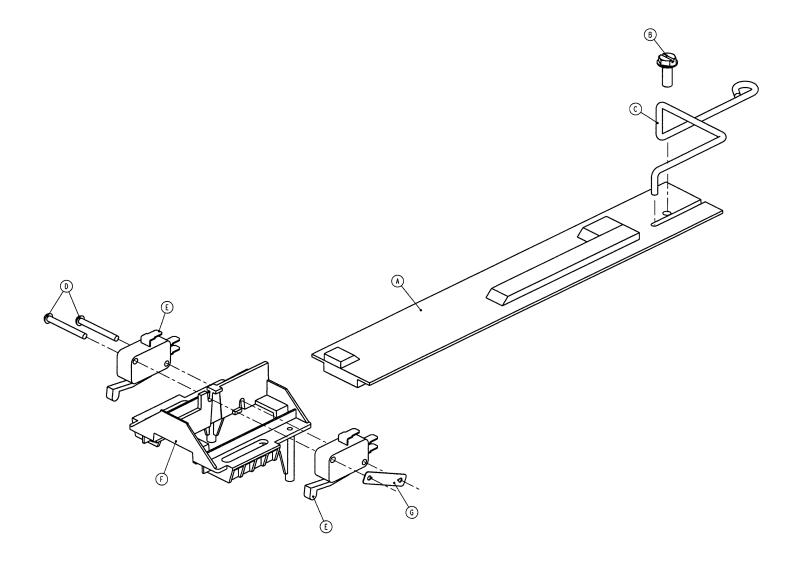



ltem	Part No.	Part Name	Qty.
А	3001–342–31	Fowler Cam Card	1
В	23–82	Hex Washer Hd. Screw	1
С	3001-300-37	Link Wire	1
D	2–3	Round Hd. Screw	2
E	3000-300-41	Micro Switch	2
F	3000-300-44	Cam Guide	1
G	16–69	Twin Fastener	1

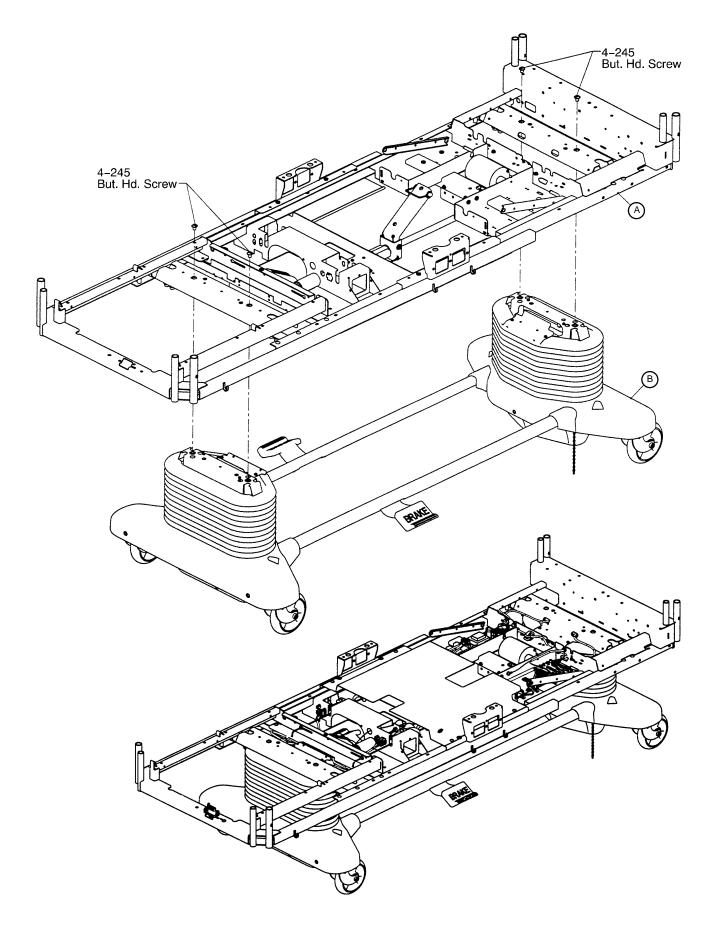


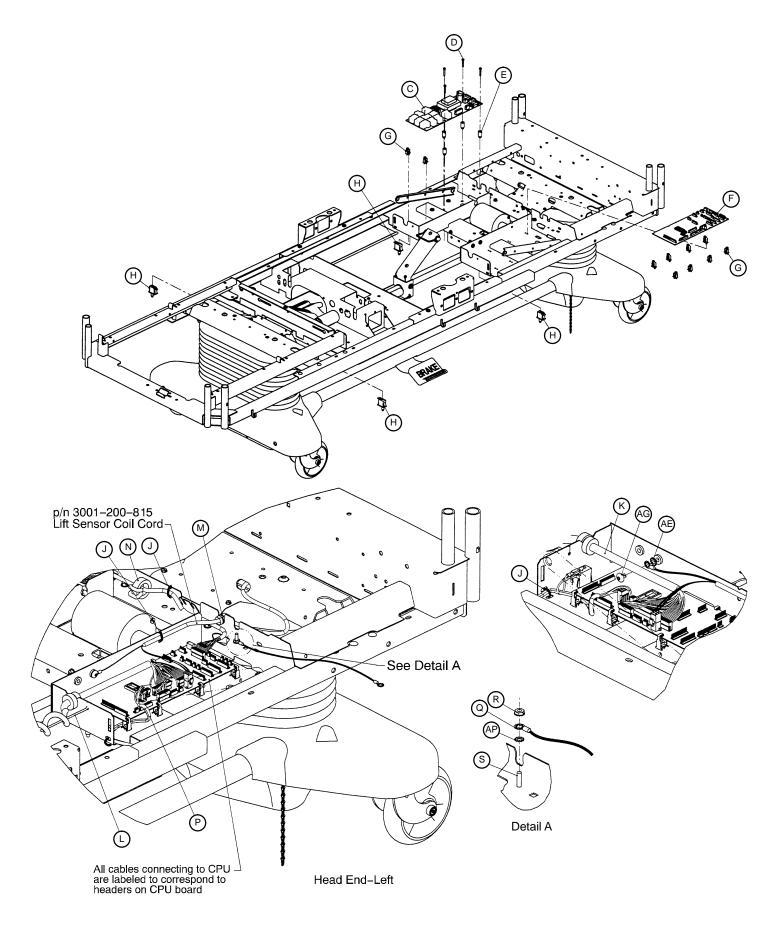


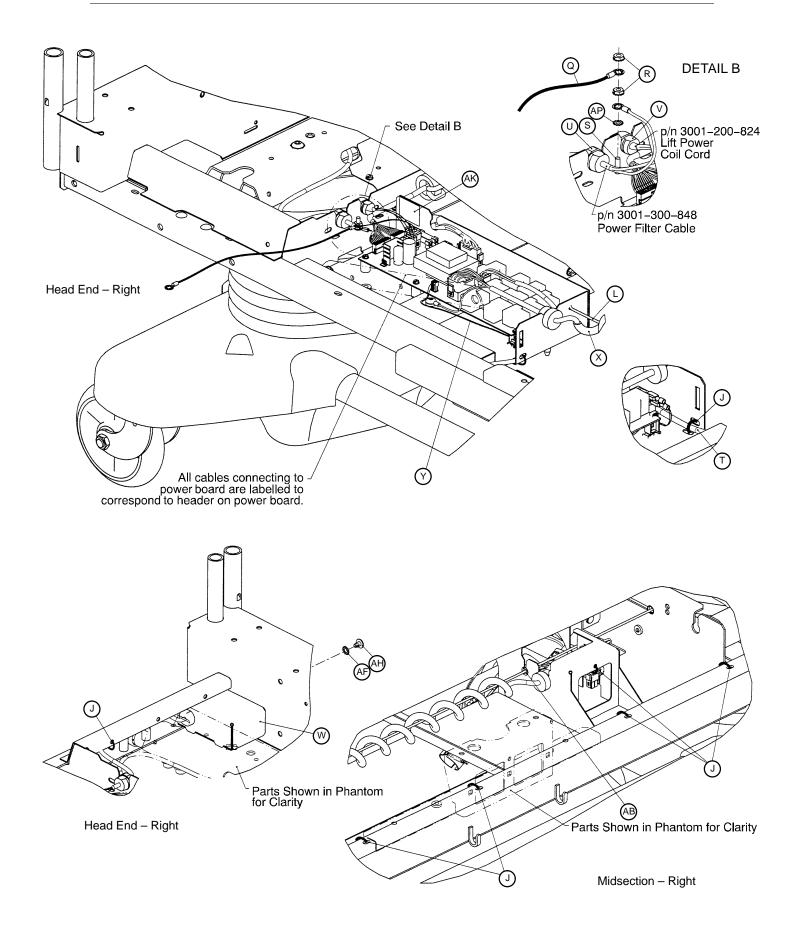

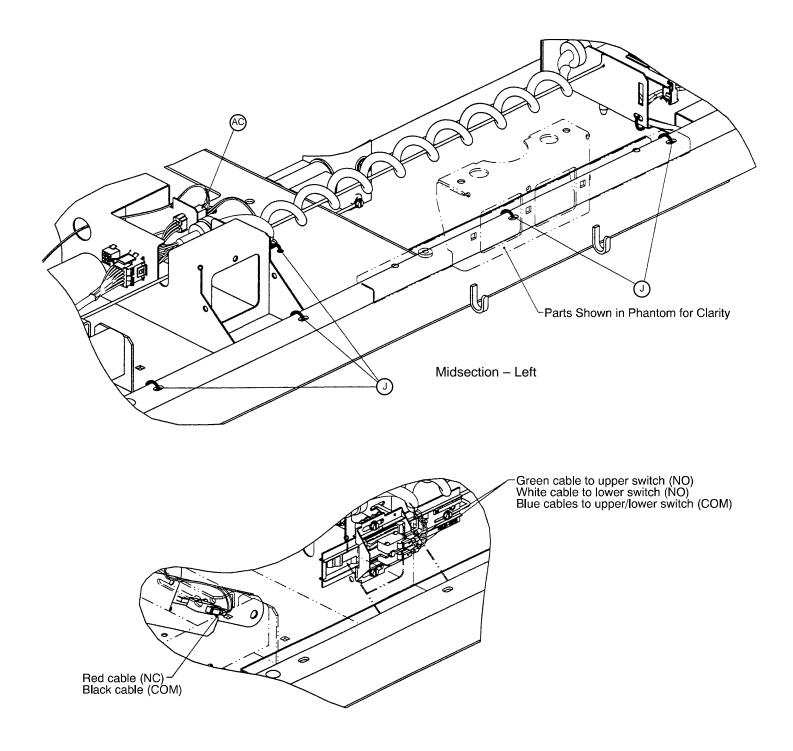

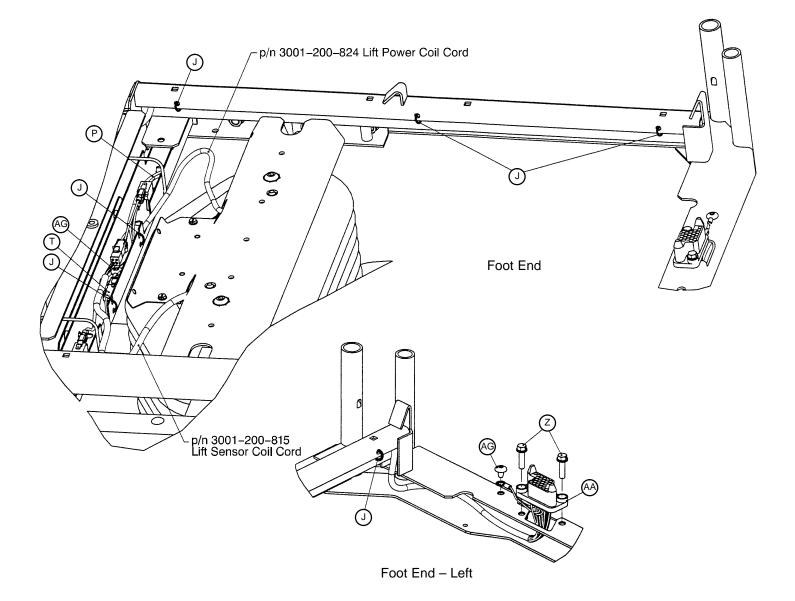

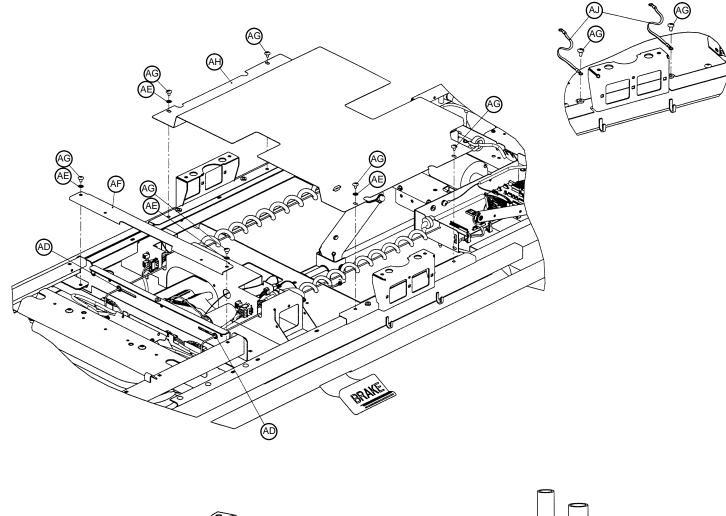

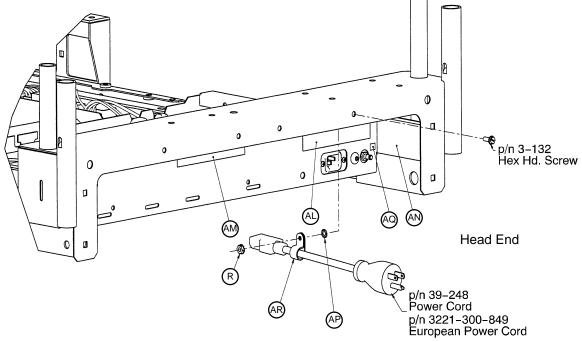


ltem	Part No.	Part Name	Qty.	ltem	Part No.	Part Name	Qty.
А	(page 115-118)	Litter Ass'y, Mechanical	1	W	(page 133)	Power Plug Ass'y	1
В	(page 86–92)	Base Assembly	1	Х	3001–300–854	Litter Power Coil Cord	1
С	3001–300–930	Power Board Assembly	1	Y	3001-300-810	Motion Int. Switch Cable	1
	3221-300-930	230V Power Board Ass'y	1	Z	3001-200-228	Mounting Standoff	2
D	3–137	Hex Washer Hd. Screw	4	AA	3001-300-833	Litter Drawer Cable	1
Е	52–750	Grommet	4	AB	52–765	Nyliner Bearing	2
F	*	CPU Board	1	AC	3001-300-829	Limit Switch Cable	1
G	59–745	Board Mount	10	AD	52–767	Nyliner Bearing	2
Н	3000-300-58	Switch Plunger	4	AE	13–10	Ext. Tooth Lock Washer	6
J	3000-300-113	Cable Tie	20	AF	3001-300-22	Cross Support Cover	1
K	3001-300-856	Litter Signal Coil Cord	1	AG	7–53	Truss Hd. Torx Screw	14
L	3001–300–11	Guide Rod	2	AH	3001–300–85	Mid Litter Cover Ass'y	1
Μ	30–47	Heyco Strain Relief	1	AJ	3001–300–870	8" Litter Ground Jumper	4
Ν	3001–300–841	PCB/CPU Ctrl. Cable	1	AK	3001–300–601	Power Fuse Label	1
Р	3001–300–845	Litter Pot Cable	1		2026–32–67	230V Power Fuse Label	1
Q	3001-300-872	21" Litter Ground Jumper	2	AL	3000-300-625	Specification Label	1
R	16–41	Kep Nut	4		3221–300–625	230V Spec. Label	1
S	3–132	Hex Washer Hd. Screw	2	AM	1550–90–1	Hosp. Grade Plug Label	1
Т	3001–300–847	Foot End Lift Cable	1	AN	2011–1–104	Warning Label	1
U	30–45	Strain Relief	1	AP	13–18	Ext. Tooth Lock Washer	3
V	30–48	Strain Relief	1	AQ	36–115	Ground Label	1
				AR	34–22	Cable Clamp	1

Electrical Litter Assembly – Standard Components


* 3001–303–770 – CPU Board (No Scale or Bed Exit)
3001–307–770 – CPU Board (Scale Only)
3001–308–770 – CPU Board (Bed Exit Only)
3001–307–772 – CPU Board with Scale and Bed Exit (All Options)


Litter Assembly – 8" Shorter Bed			Litter Assembly – Standard Bed				
ltem	Part No.	Part Name	Qty.	ltem	Part No.	Part Name	Qty.
А	(page 115–118)	Litter Ass'y, Mechanical	1	А	(page 115–118)	Litter Ass'y, Mechanical	1


Power Plug Assembly

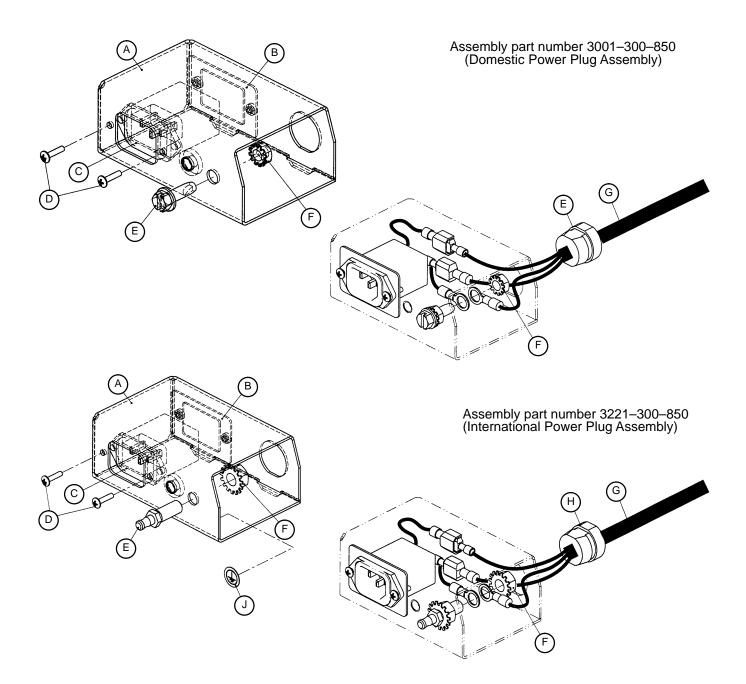


ltem	Part No.	Part Name	Qty.
А	3001–300–18	Plug Box	1
В	3001–300–17	Plug Mounting Bracket	1
С	3001–300–840	Power Plug	1
D	7–46	Phillips Truss Hd. Screw	2
E	3–226	H. Washer Hd. Scr. (Domestic)	1
	2011–1–215	Grounding Lug (International)	1
F	16–6	Kep Nut (Domestic)	2
	16–33	Kep Nut (International)	2
G	3001–300–848	Filter Power Cable	1
Н	30–45	Strain Relief	1
J	36–46	Ground Label (International)	1

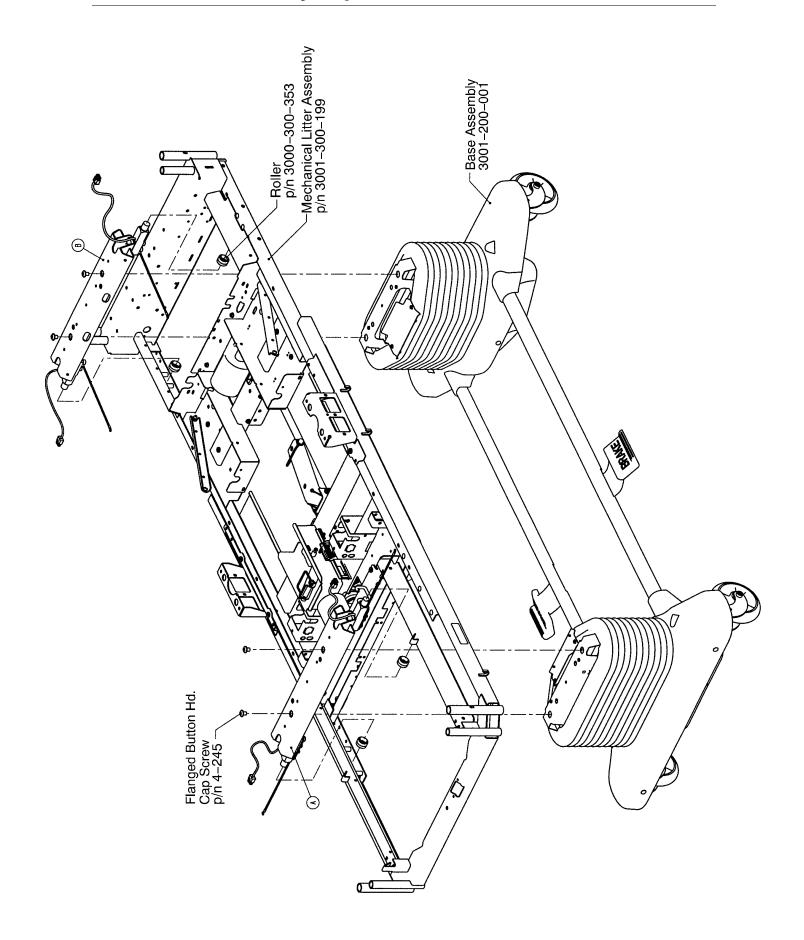

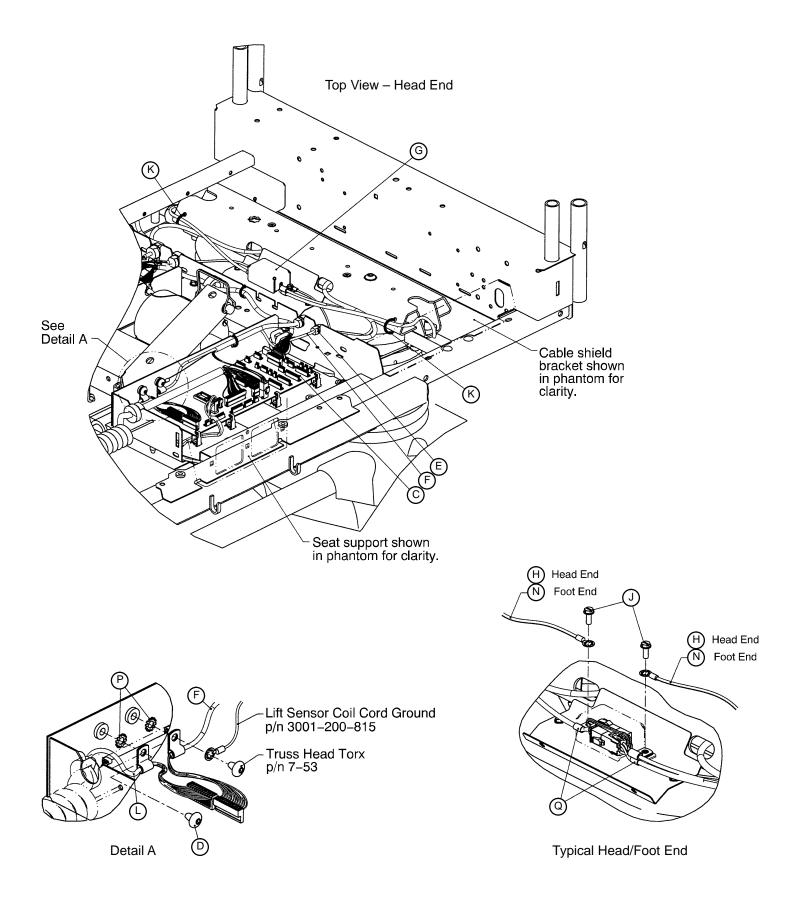

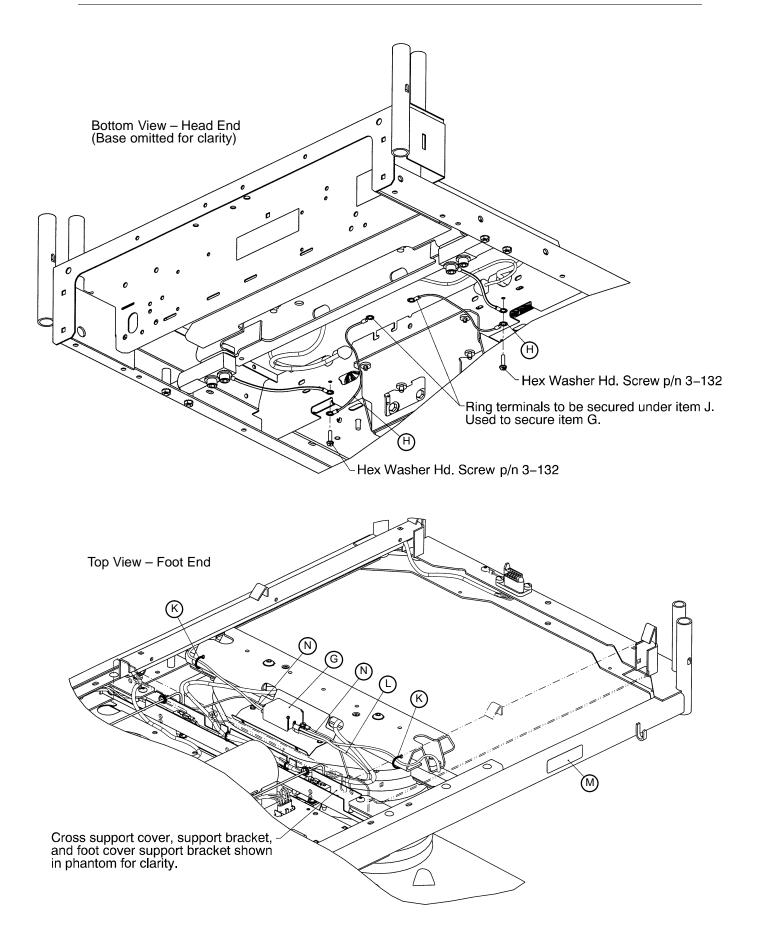

Litter Assembly, Optional Scale and/or Bed Exit

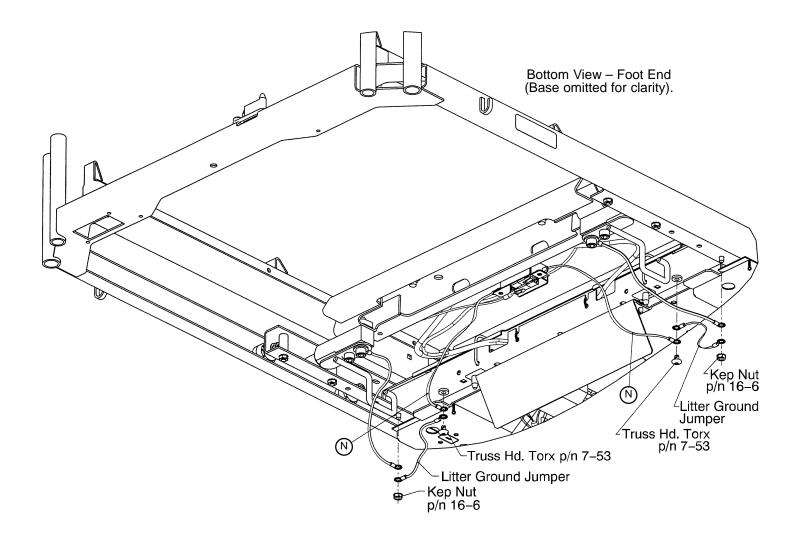


Litter Assembly, Optional Scale and/or Bed Exit


ltem	Part No.	Part Name	Qty.
А	(page 120 & 121)	Lift Header Ass'y, Scale w/Foley	1
В	(page 120 & 121)	Lift Header Ass'y, Scale	1
С	*	CPU Board Assembly	1
D	7–53	Truss Head Screw	1
E	30–49	Strain Relief	1
F	3001-307-822	Head End Load Cell Cable	1
G	3001–307–13	Load Cell Connector Cover	2
Н	3001-300-870	8" Ground Jumper	2
J	3–224	Hex Washer Head Screw	4
K	3000–300–113	Cable Tie	4
L	3001-307-823	Foot End Load Cell Cable	1
Μ	3000–300–601	Foley Bag Hook Label	2
Ν	3001-300-871	11" Ground Jumper	2
Р	13–10	External Tooth Lock Washer	1
Q	59–768	Cable Clamp	4

* 3001–303–780 CPU Board (No Scale or Bed Exit) 3001–307–783 CPU Board (Scale and Bed Exit)

Optional 110V Outlet Box Assembly

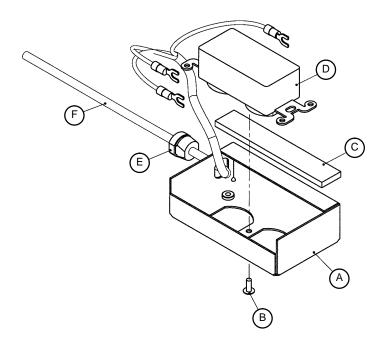

ltem	Part No.	Part Name	Qty.
А	3001-320-20	110V Enclosure Bracket	1
В	7–28	Phillips Truss Hd. Screw	1
С	7900–1–291	Foam Tape	1
D	59–733	Hospital Grade Receptacle	1
E	34–133	Strain Relief	1
F	3001–320–801	110V Outlet Cable	1

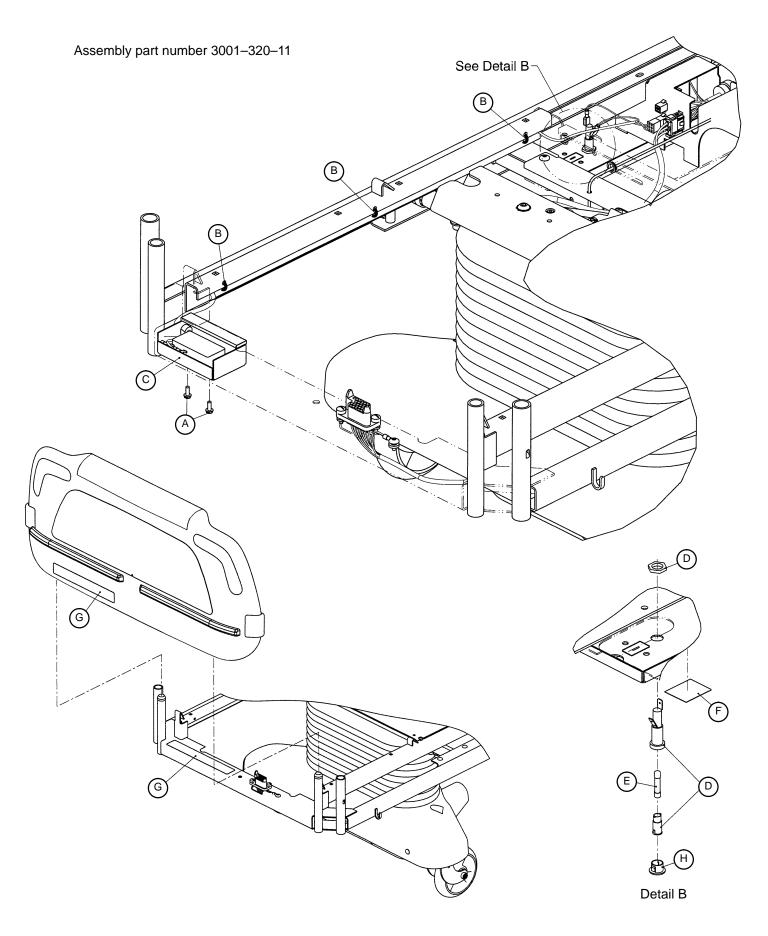


Litter Assembly, Optional 110V Outlet

ltem	Part No.	Part Name	Qty.
А	3–224	Hex Washer Hd. Screw	2
В	3000–300–113	8" Cable Tie	3
С	(page 139)	110V Outlet Box	1
D	2020-2-664	Fuse Holder	1
E	59–770	3 Amp Fuse	1
F	3001–320–606	Fuse Label	1
G	3001–320–616	110V Outlet Label	2

Litter Assembly – No 110V Outlet Option

ltem	Part No.	Part Name	Qty.
Н	59–735	Heyco Plug	1

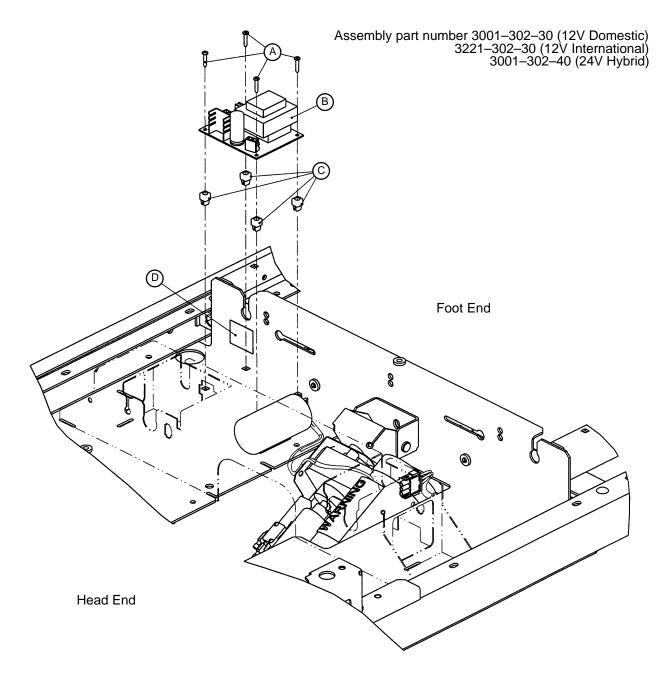



12V DMS Litter Option Assembly

ltem	Part No.	Part Name	Qty.
А	23–108	Phillips Hd. Screw	4
В	3001-302-910	DMS Power Supply	1
	3221–302–910	Intnat'll DMS Power Supp	ly 1
С	52–287	Grommet	4
D	3001–302–601	Fuse Warning Label	1
	3221–302–601	Intnat'l Fuse Warning Lab	el 1
Е	3000–300–113	8" Cable Tie	5
F	59–725	Female Screwlock	2
G	3001-302-812	DMS Port Cable	1
Н	7–53	Truss Hd. Screw	1
J	59–771	Static Plug	1


24V DMS Litter Option Assembly

ltem	Part No.	Part Name	Qty.
А	23–108	Phillips Hd. Screw	4
В	3000-302-939	DMS Power Supply	1
С	52–287	Grommet	4
D	2020-250-527	Fuse Warning Label	1
Е	3000-300-113	8" Cable Tie	5
F	7–46	Truss Hd. Screw	3
G	3001-302-832	24V Hybrid DMS Cable	1
н	7–53	Truss Hd. Screw	1
J	59–771	Static Plug	1

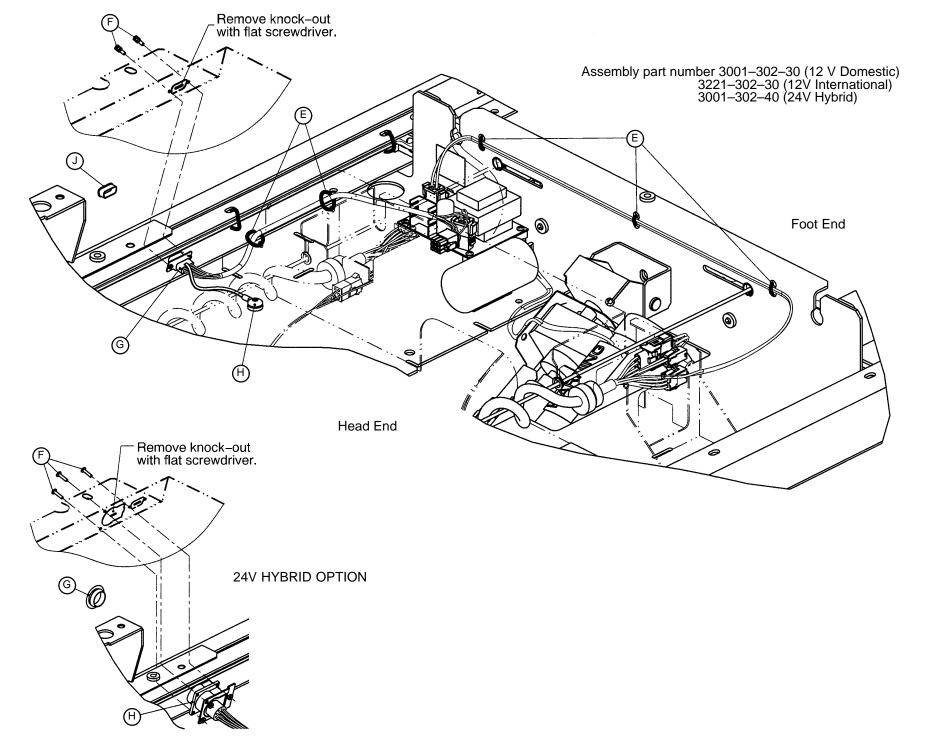


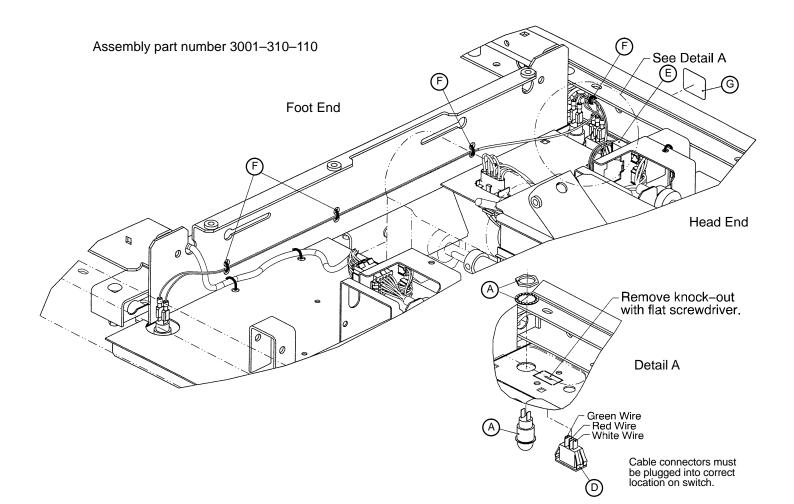

Item	Part No.	Part Name	Qty.
А	3000–310–830	Night Light Assembly	2
D	3000-310-828	Night Light Switch	1
E	3001–310–801	Night Light Cable	1
F	3000-300-113	8" Cable Tie	4
G	3000-300-617	Night Light Label	1
N/A	59–798	Night Light Bulb	1

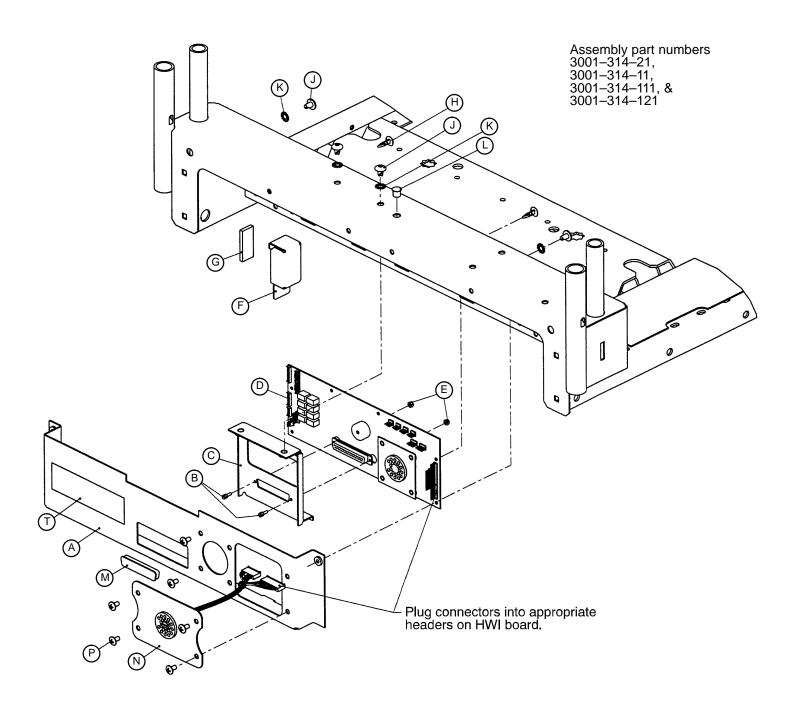

Litter Assembly, Optional HWI w/Comm. and Ports

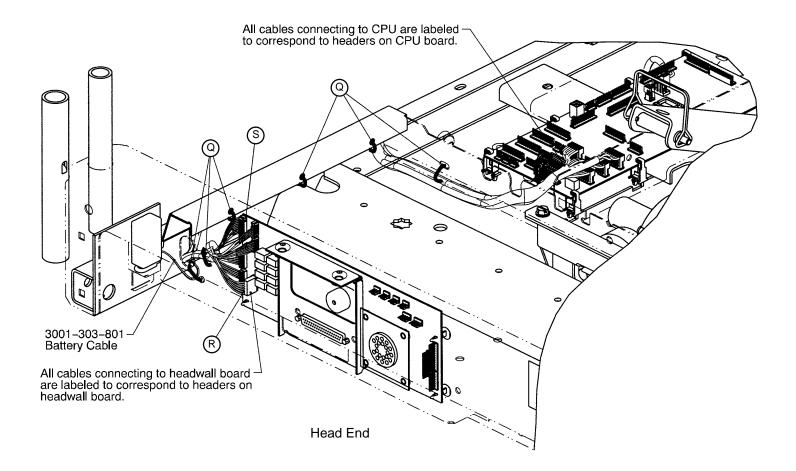

3001–314–21 HWI w/2 Stryker Ports


ltem	Part No.	Part Name	Qty.	ltem	Part No.	Part Name	Qty.
А	3001–340–41	Enclosure, HWI	1	А	3001–340–41	Enclosure, HWI	1
В	59–727	Jack Screw	2	В	59–727	Jack Screw	2
С	3001–303–11	HWI PCB Bracket	1	С	3001–303–11	HWI PCB Bracket	1
D	3001-314-900	HWI PCB w/Stryker Port	1	D	3001-314-900	HWI PCB w/Stryker Port	1
Е	16–7	Fiberlock Nut	2	Е	16–7	Fiberlock Nut	2
F	(page 148)	Battery Enclosure	1	F	(page 148)	Battery Enclosure	1
G	7900-1-291	Foam Tape	1	G	7900-1-291	Foam Tape	1
Н	3000-300-115	Standoff	4	Н	3000-300-115	Standoff	4
J	7–53	Truss Head Screw	4	J	7–53	Truss Head Screw	4
K	13–10	External Tooth Lock Washer	r 4	K	13–10	External Tooth Lock Washer	4
L	59–738	Dome Plug	1	L	59–738	Dome Plug	1
М	59–710	Static Plug	1	Μ	59–710	Static Plug	1
Ν	3001-314-802	Port Cable Assembly	1	Ν	3001–314–899	Custom Port Cable Ass'y	1
Р	7–19	Truss Head Screw	6	Р	7–19	Truss Head Screw	6
Q	3000-300-113	8" Cable Tie	6	Q	3000-300-113	8" Cable Tie	6
R	3001-303-810	HWI to CPU Cable	1	R	3001-303-810	HWI to CPU Cable	1
S	3001–314–811	Pendant Port Cable	1	S	3001–314–811	Pendant Port Cable	1
Т	1550–90–1	Caution Label	1	Т	1550–90–1	Caution Label	1

3001–314–11 HWI w/1 Stryker Port


3001-314-111 HWI w/1 Custom Port

ltem	Part No.	Part Name	Qty.	ltem	Part No.	Part Name	Qty.
А	3001-340-41	Enclosure, HWI	1	А	3001–340–61	Enclosure, HWI w/Cust. Port	1
В	59–727	Jack Screw	2	В	59–727	Jack Screw	2
С	3001–303–11	HWI PCB Bracket	1	С	3001–303–11	HWI PCB Bracket	1
D	3001-314-900	HWI PCB w/Stryker Port	1	D	3001-303-900	HWI PCB	1
Е	16–7	Fiberlock Nut	2	Е	16–7	Fiberlock Nut	2
F	(page 148)	Battery Enclosure	1	F	(page 148)	Battery Enclosure	1
G	7900-1-291	Foam Tape	1	G	7900-1-291	Foam Tape	1
Н	3000-300-115	Standoff	4	Н	3000-300-115	Standoff	4
J	7–53	Truss Head Screw	4	J	7–53	Truss Head Screw	4
K	13–10	External Tooth Lock Washe	r 4	K	13–10	External Tooth Lock Washer	4
L	59–738	Dome Plug	1	L	59–738	Dome Plug	1
М	59–710	Static Plug	1	Μ	59–710	Static Plug	1
Р	7–19	Truss Head Screw	4	Ν	3001–314–899	Custom Port Cable Ass'y	1
Q	3000-300-113	8" Cable Tie	6	Р	7–19	Truss Head Screw	4
R	3001-303-810	HWI to CPU Cable	1	Q	3000-300-113	8" Cable Tie	6
S	3001-314-811	Pendant Port Cable	1	R	3001-303-810	HWI to CPU Cable	1
Т	1550–90–1	Caution Label	1	S	3001–314–811	Pendant Port Cable	1
				Т	1550–90–1	Caution Label	1

3001-314-121 HWI w/1 Stryker & 1 Custom Port

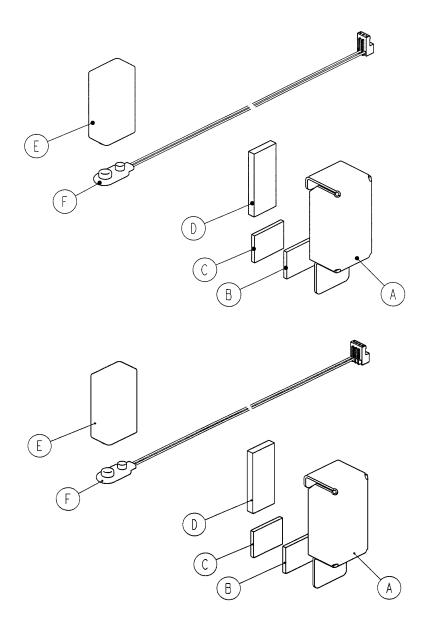


Item	Part No.	Part Name	Qty.
А	3001-303-12	Battery Enclosure	1
В	29–10	Fine Duo–Lock	1
С	29–8	Coarse Duo–Lock	1
D	7900–1–291	Foam Tape	1
E	3000-303-871	Battery	1
F	3001–303–801	Battery Cable	1
F	3001–303–811	Battery Cable	1

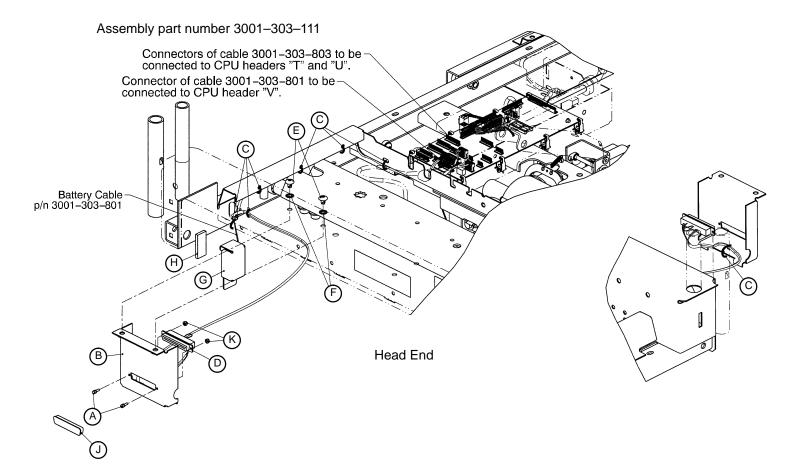

Item	Part No.	Part Name	Qty.
А	59–727	Jack Screw	2
В	3001–303–16	Enclosure, HWI Nurse Call Only	1
С	3000-300-113	8" Cable Tie	7
D	3001–303–813	Cable, HWI Nurse Call Only	1
Е	7–53	Truss Hd. Screw	2
F	13–10	External Tooth Lock Washer	2
G	(page 148)	Battery Enclosure Assembly	1
K	7900–1–291	Foam Tape	1
Ν	59–710	Static Plug	1
Р	16–7	Fiberlock Nut	2

Litter Ass'y, Optional HWI Communication Capability

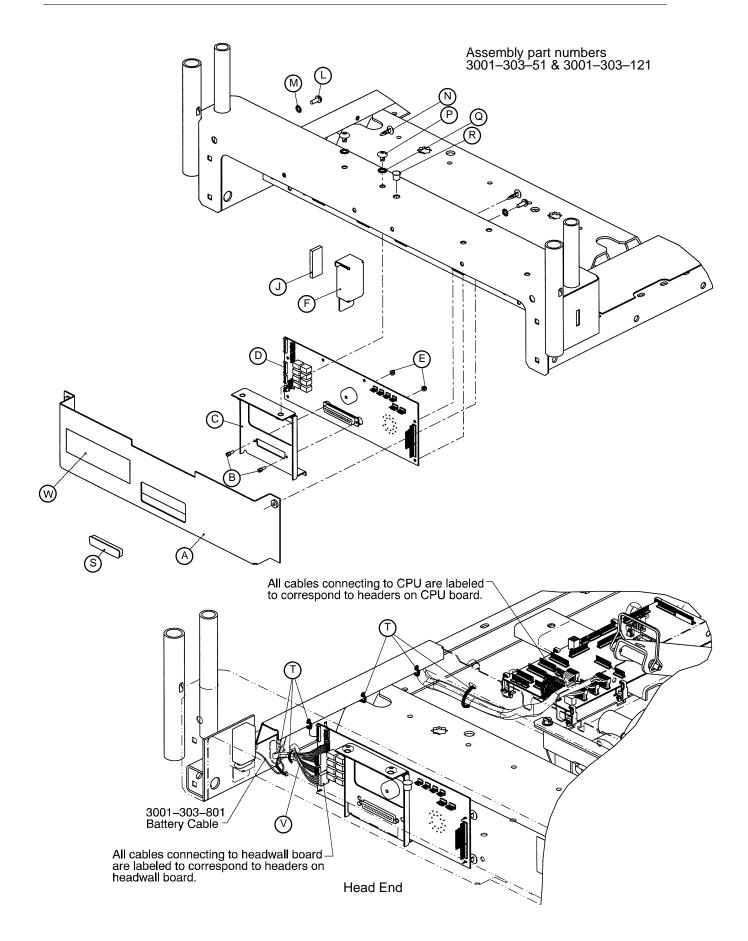


3001–303–51 Nurse Call & Comm. Capability

ltem	Part No.	Part Name	Qty.
А	3001-340-41	HWI Standard Enclosure	e 1
В	59–727	Jack Screw	2
С	3001–303–11	HWI PBC Bracket	1
D	3001-303-900	HWI PCB Assembly	1
Е	16–7	Fiberlock Nut	2
F	(see below)	Battery Enclosure	1
J	7900–1–291	Foam Tape	1
L	3–224	Hex Washer Hd. Screw	2
Μ	13–18	Ext. Tooth Lock Washer	2
Ν	3000–300–115	Standoff	4
Р	7–53	Truss Hd. Screw	2
Q	13–10	Ext. Tooth Lock Washer	2
R	59–738	Dome Plug	1
S	59–710	Static Plug	1
Т	3000–300–113	8" Cable Tie	6
V	3001–303–810	HWI to CPU Cable	1
W	1550–90–1	Caution Label	1

3001–303–121 Communication Capabitlty


ltem	Part No.	Part Name	Qty.
А	3001-340-41	HWI Standard Enclosure	e 1
В	59–727	Jack Screw	2
С	3001–303–11	HWI PCB Bracket	1
D	3001-303-900	HWI PCB Assembly	1
Е	16–7	Fiberlock Nut	2
L	3–224	Hex Washer Hd. Screw	2
М	13–18	Ext. Tooth Lock Washer	2
Ν	3000-300-115	Standoff	4
Р	7–53	Truss Hd. Screw	2
Q	13–10	Ext. Tooth Lock Washer	2
R	59–738	Dome Plug	1
S	59–710	Static Plug	1
Т	3000-300-113	8" Cable Tie	5
V	3001–303–810	HWI to CPU Cable	1
W	1550–90–1	Caution Label	1

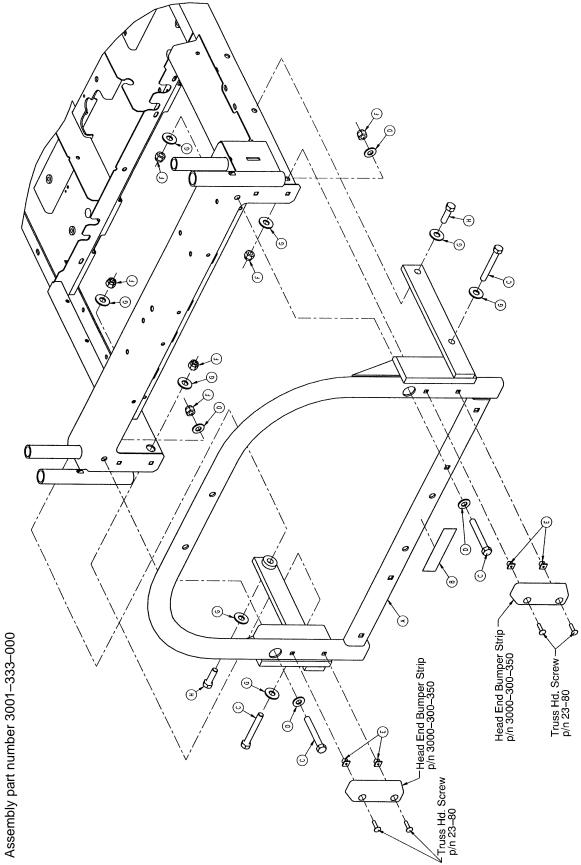


Litter Assembly, Optional Accessory Adapter Frame

ltem	Part No.	Part Name	Qty.
А	3001–333–10	Frame Assembly	1
В	3000–333–15	Accessory Adapter Label	1
С	3–208	Hex Hd. Cap Screw	4
D	11–301	Flat Washer	4
E	3000-300-2	Plastic Clip Nut	4
F	16–35	Nylock Nut	6
G	11–361	Flat Washer	8
Н	3–120	Hex Hd. Cap Screw	2


Assembly part number 3001–334–10

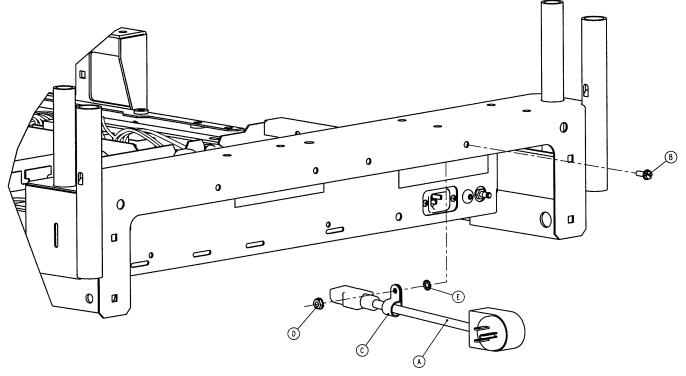


ltem	Part No.	Part Name	Qty.
А	3001–334–850	90° Power Cord (12 O'Clock)	1
	3001–334–851	90° Power Cord (9 O'Clock)	1
	3001–334–852	90° Power Cord (6 O'Clock)	1
	3001–334–853	90° Power Cord (3 O'Clock)	1
В	3–132	Hex Washer Hd. Screw	1
С	34–22	Cable Clamp	1
D	16–41	Kep Nut	1
E	13–18	External Tooth Lock Washer	1

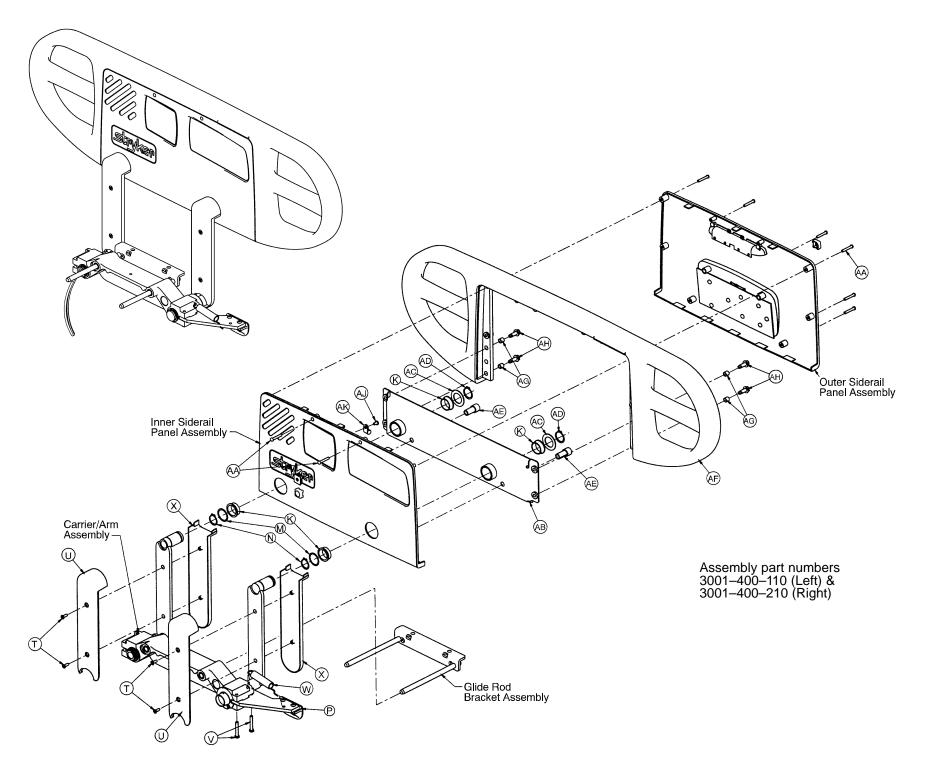




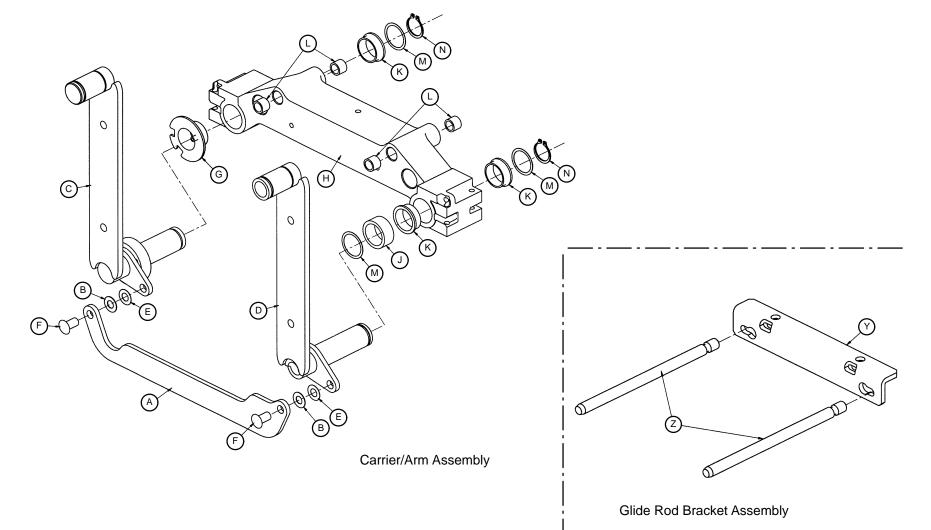



Head End Siderail Assembly

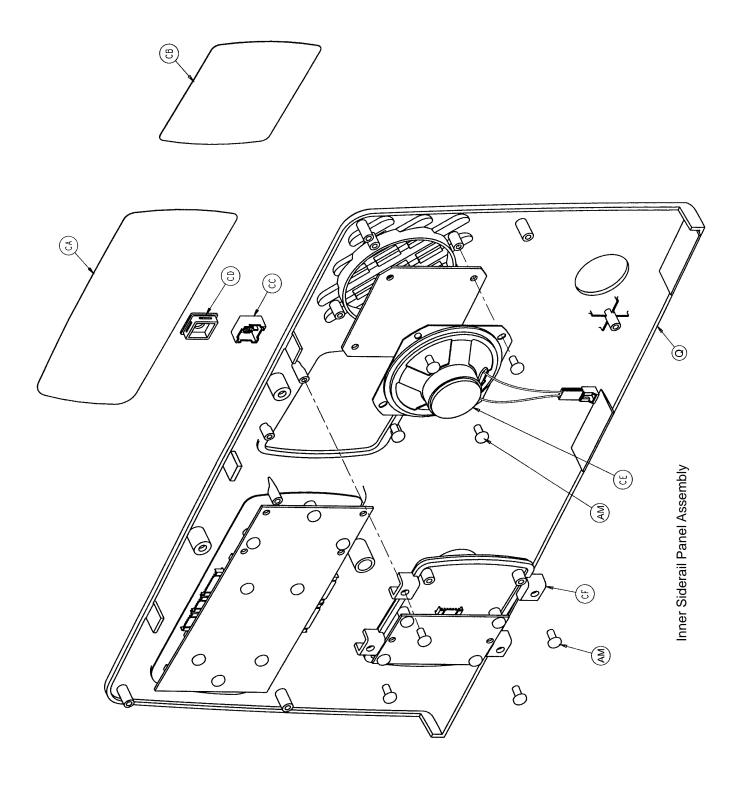

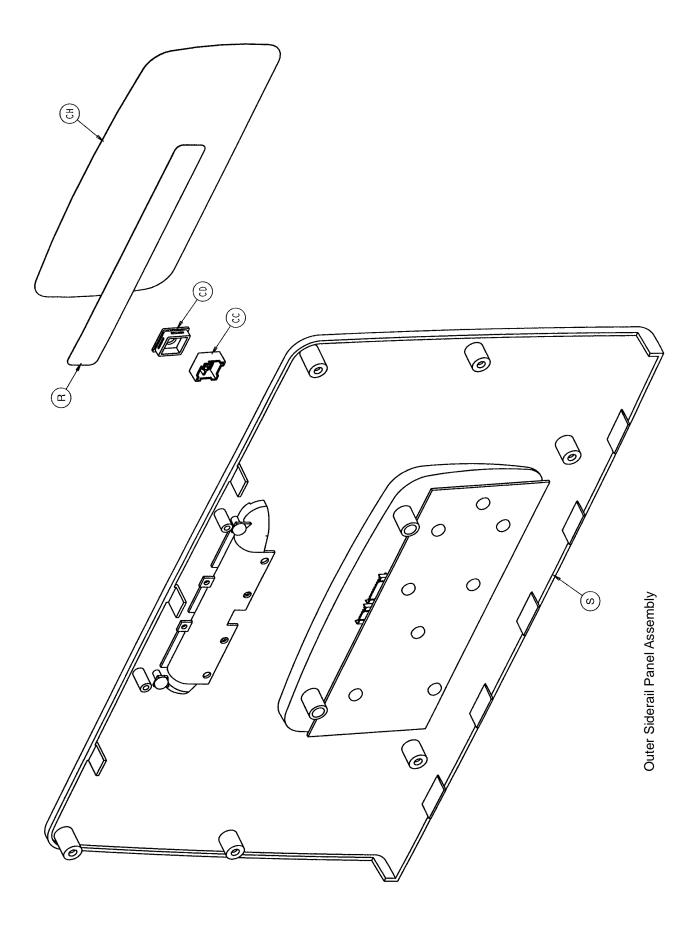

Head End Siderail Assembly

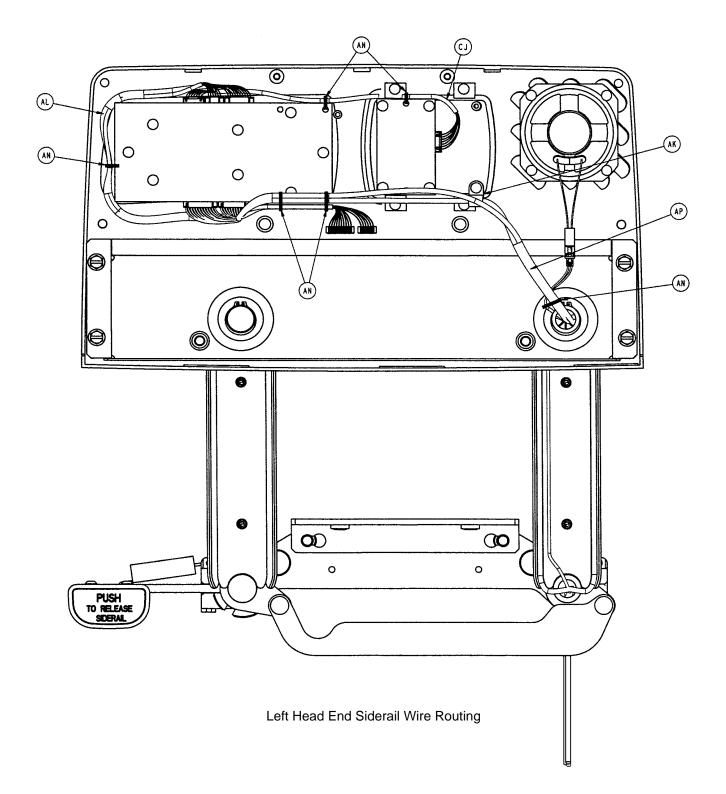

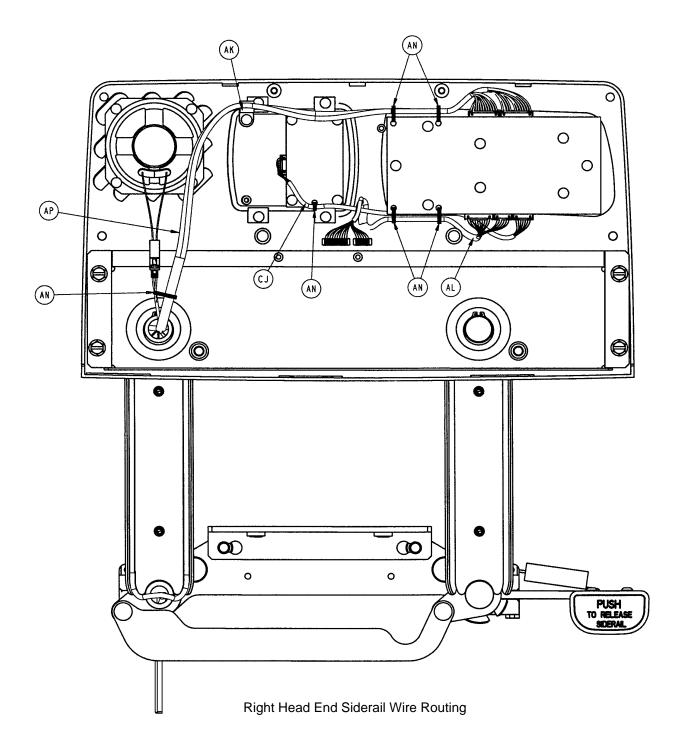

3001–400–110 Left Standard Components

А

В

С

D

Е

F

G

Н

J

Κ

L

Μ

Ν

Ρ

Q

R

S

Т

U

V

W

Х

Υ

Ζ

AA

AB

AC AD

AE

AF

AG

AH

AJ

AK

AL

AM

AN

Item Part No. Part Name Part No. Part Name Qty. ltem Qty. 3001-400-11 Timing Link Weldment, Hd. 1 А 3001-400-11 Timing Link Weldment, Hd. 1 11 - 403Shim Washer В 11-403 Shim Washer 2 2 3001-400-127 Arm Weldment, Left, Foot 1 С 3001-400-227 Arm Weldment, Right, Foot. 1 D 3001-400-128 Arm Weldment, Left, Head 1 3001-400-228 Arm Weldment, Right, Head 1 Е Nylon Washer 11-377 Nylon Washer 2 11-377 2 Siderail Linkage Rivet 2 F 3001-400-501 3001-400-501 Siderail Linkage Rivet 2 G 3001-400-513 Wear Bushing 1 3001-400-513 Wear Bushing 1 Siderail Carrier Н Siderail Carrier 3001-400-528 1 3001-400-528 1 3001-400-530 Siderail Arm Spacer 1 J 3001-400-530 Siderail Arm Spacer 1 7 3000-400-513 Flange Bearing Κ 3000-400-513 Flange Bearing 7 3000-400-557 Sleeve Bearing 4 3000-400-557 Sleeve Bearing 4 11-353 Shim Washer 5 Μ 11-353 Shim Washer 5 28-128 **Retaining Ring** 4 Ν 28-128 Retaining Ring 4 Ρ (page 185) Release Lever, Left 1 (page 186) Release Lever, Right 1 Q (page 162) Inner Panel Ass'y, Left 1 (page 162) Inner Panel Ass'y, Right 1 3000-400-556 3000-400-556 Warning Label 1 R Warning Label 1 Outer Panel Ass'y S Outer Panel Ass'y (page 163) 1 (page 163) 1 Self-Tapping Screw 4 Т Self-Tapping Screw 23-261 23-261 4 5000-20-5 Inner Arm Cover 2 U 5000-20-5 Inner Arm Cover 2 2 3-330 Hex Washer Hd. Screw V 3-330 Hex Washer Hd. Screw 2 3000-200-334 Extension Spring 1 W 3000-200-334 Extension Spring 1 Outer Arm Cover 2 Outer Arm Cover 2 5000-20-6 Х 5000-20-6 Υ 3001-400-553 Mounting Bracket 1 3001-400-553 Mounting Bracket 1 2 2 3001-400-554 Glide Rod Ζ 3001-400-554 Glide Rod 23-90 High-Low Tapping Screw 8 AA High-Low Tapping Screw 8 23-90 3001-400-130 Supt. Weldment, Hd., Lt. 3001-400-230 Supt. Weldment, Hd., Rt. 1 AB 1 11-343 Shim Washer 2 AC 11-343 Shim Washer 2 28-132 Bowed Retaining Ring 2 AD 28-132 Bowed Retaining Ring 2 3000-400-523 Panel Spacer 2 Panel Spacer 2 AE 3000-400-523 3001-400-515 Head Rail 1 AF 3001-400-515 Head Rail 1 3001-400-558 Siderail Spacer 4 AG 3001-400-558 Siderail Spacer 4 3-226 Hex Washer Hd. Screw 4 AH 3-226 Hex Washer Hd. Screw 4 High-Low Tapping Screw 23-86 High-Low Tapping Screw 1 AJ 23-86 1 3000–300–478 CPR Conduit Clamp 1 AK 3000-300-478 **CPR** Conduit Clamp 1 3001-400-802 Main Outside Cable, Lt. 1 AL 3001-400-801 Main Outside Cable, Rt. 1 High-Low Tapping Screw 8 AM 23-112 High-Low Tapping Screw 8 23-112 3000-300-114 4" Cable Tie 4 AN 3000-300-114 4" Cable Tie 5

3001–443–110 Left Rail – Standard Bed

ltem	Part No.	Part Name	Qty.
AP	3001–300–843	Siderail Harness, Lt.	1

3001–442–110 Left Rail – 8" Shorter Bed

ltem	Part No.	Part Name	Qty.
AP	3001-342-843	Siderail Harness, Lt.	1

3001–443–210 Right Rail – Standard Bed


3001–400–210 Right Standard Components

ltem	Part No.	Part Name	Qty.
AQ	3001–300–842	Siderail Harness, Rt.	1


3001-442-210 Right Rail - 8" Shorter Bed

ltem	Part No.	Part Name	Qty.
AQ	3001-342-842	Siderail Harness, Rt.	1

Siderail Inner Panel Assembly

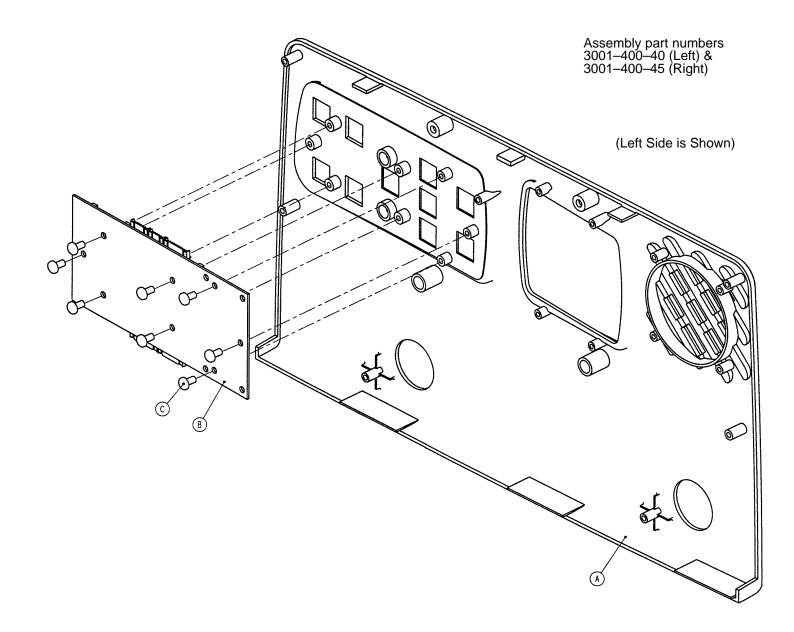

ltem	Part No.	Part Name	Qty.
А	3001-400-101	Left Inner Panel	1
	3001-400-201	Right Inner Panel	1
В	3001-400-900	Inner Siderail PCB Assembly	1
С	23–112	Hi–Low Tapping Screw	8

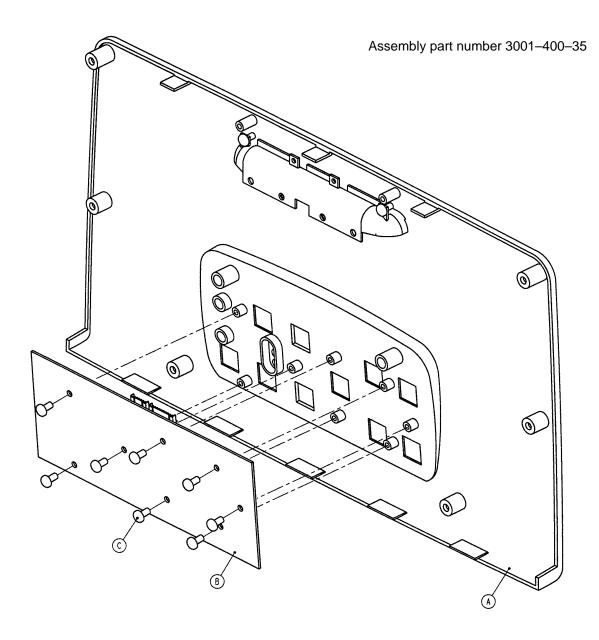


Item	Part No.	Part Name	Qty.
А	3001-400-50	Outer Panel Assembly	1
В	3001-400-910	Outer Siderail PCB Assembly	1
С	23–112	Hi–Low Tapping Screw	8

1


Standard Siderail

ltem	Part No.	Part Name	
CA	3001-400-620	Standard Label, Inner, Left	
CA	3001–400–610	Standard Label, Inner, Right	
CC	3001-400-953	Switch Cap	
CD	3001-400-522	Filler Cap	
CE	3001–400–517	Speaker Seal	
CF	3001–400–535	Blank Module	
CH	3001-400-630	Standard Label, Outer, Left	
СН	3001–400–640	Standard Label, Outer, Right	


Inner Right Siderail Label

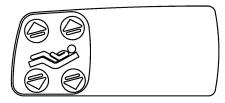


Outer Right Siderail Label

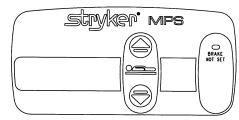


Inner Right Siderail Label

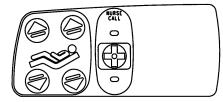

Outer Right Siderail Label

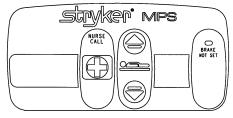

Inner Right Siderail Label

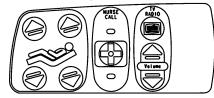

Outer Right Siderail Label



Standard Siderail with Nurse Call

ltem	Part No.	Part Name	Qty.
CA	3001–400–621	Nurse Call Label, Inner, Left	1
CA	3001–400–611	Nurse Call Label, Inner, Right	1
CC	3001-400-953	Switch Cap	16
CD	3001-400-522	Filler Cap	24
CE	3000–403–831	Speaker With Cable	2
CF	3001–400–535	Blank Module	2
CH	3001–400–631	Nurse Call Label, Outer, Left	1
CH	3001–400–641	Nurse Call Label, Outer, Right	1


Standard Siderail with Nurse Call & TV/Radio

ltem	Part No.	Part Name	Qty.
CA	3001–400–622	NC, TV Label, Inner, Left	1
CA	3001–400–612	NC, TV Label, Inner, Right	1
CC	3001-400-953	Switch Cap	22
CD	3001-400-522	Filler Cap	18
CE	3000-403-831	Speaker With Cable	2
CF	3001–400–535	Blank Module	2
CH	3001–400–631	Nurse Call Label, Outer, Left	1
CH	3001–400–641	Nurse Call Label, Outer, Right	1


Standard Siderail with Nurse Call, TV/Radio & Lights

ltem	Part No.	Part Name	Qty.
CA	3001-400-623	NC, TV, Lights Label, Inner, Left	1
CA	3001–400–613	NC, TV, Lights Label, Inner, Right	1
CC	3001-400-953	Switch Cap	26
CD	3001-400-522	Filler Cap	14
CE	3000–403–831	Speaker with Cable	2
CF	3001–400–535	Blank Module	2
CH	3001–400–631	Nurse Call Label, Outer, Left	1
СН	3001-400-641	Nurse Call Label, Outer, Right	1

Inner Right Siderail Label

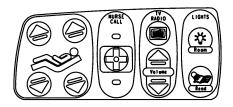


Outer Right Siderail Label

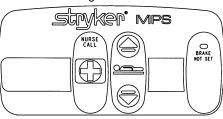

Standard Siderail w/Nurse Call, TV/Radio, Lights, & DMS

ltem	Part No.	Part Name	Qty.
AN	3000-300-114	Cable Tie	3
CA	3001-400-623	NC, TV, Lights, DMS Label, In., Lt.	1
CA	3001-400-613	NC, TV, Lights, DMS Label, In., Rt.	1
CC	3001-400-953	Switch Cap	20
CE	3000–403–831	Speaker With Cable	2

Inner Right Siderail Label

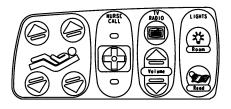


Std. Siderail w/Nurse Call, TV/Radio, Lights, & DMS Auto/Man.


ltem	Part No.	Part Name	Qty.
СВ	3001-402-1	DMS Label, Left	1
СВ	3001-402-2	DMS Label, Right	1
CC	3001-400-953	Switch Cap	12
CD	3001–400–522	Filler Cap	12
CF	3001-402-30	DMS Module	2
СН	3001–400–635	NC, DMS Label, Outer, Left	1
СН	3001–400–645	NC, DMS Label, Outer, Right	1
CJ	3001-402-804	Main to Options PCB Cable, Left	1
CJ	3001-402-803	Main to Options PCB Cable, Right	1

Std. Siderail w/Nurse Call, TV/Radio, Lights, & DMS Auto Only

ltem	Part No.	Part Name	Qty.
CC	3001-400-953	Switch Cap	6
CD	3001-400-522	Filler Cap	14
СН	3001-400-655	NC/DMS Label, Outer/Left	1
СН	3001–400–665	NC/DMS Label, Outer/Right	1

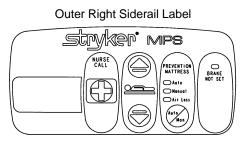



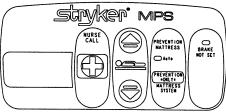

Std. Siderail w/Nurse Call, TV/Radio, Lights, DMS, & G/F

ltem	Part No.	Part Name	Qty.
AN	3000-300-114	Cable Tie	3
CA	3001–400–623	NC, TV, Lights, DMS Label, In., Lt.	1
CA	3001–400–613	NC, TV, Lights, DMS Label, In., Rt.	1
CC	3001–400–953	Switch Cap	20
CE	3000-403-831	Speaker With Cable	2

Std. Siderail w/NC, TV/Radio, Lights, G/F, & DMS Auto/Man.

ltem	Part No.	Part Name	Qty.
CB	3001-402-1	DMS Label, Left	1
CB	3001-402-2	DMS Label, Right	1
CC	3001–400–953	Switch Cap	20
CD	3001–400–522	Filler Cap	4
CF	3001-402-30	DMS Module	2
СН	3001–400–633	NC, DMS, G/F Label, Outer, Left	1
СН	3001–400–643	NC, DMS, G/F Label, Outer, Right	1
CJ	3001-402-804	Main to Options PCB Cable, Left	1
CJ	3001-402-803	Main to Options PCB Cable, Right	1


Std. Siderail w/NC, TV/Radio, Lights, G/F, & DMS Auto Only


ltem	Part No.	Part Name	Qty.
CC	3001–400–953	Switch Cap	14
CD	3001–400–522	Filler Cap	6
СН	3001–400–653	NC, DMS, G/F Label, Outer, Left	1
СН	3001-400-663	NC, DMS, G/F Label, Outer, Right	1

Standard Siderail w/Nurse Call, TV/Radio, Lights, & G/F

ltem	Part No.	Part Name	Qty.
CA	3001–400–623	NC, TV, Lights Label, Inner, Left	1
CA	3001–400–613	NC, TV, Lights Label, Inner, Right	1
CC	3001–400–953	Switch Cap	34
CD	3001-400-522	Filler Cap	6
CE	3000-403-831	Speaker with Cable	2
CF	3001–400–535	Blank Module	2
CH	3001-400-632	NC, G/F Label, Outer, Left	1
СН	3001-400-642	NC, G/F Label, Outer, Right	1

Inner Right Siderail Label

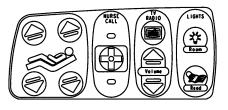


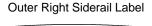







Outer Right Siderail Label

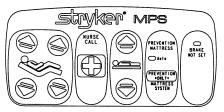

Inner Right Siderail Label

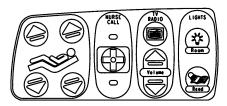

Outer Right Siderail Label

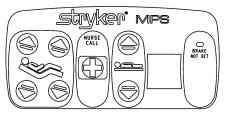

166

Standard Siderail w/Nurse Call, TV/Radio, DMS, & G/F

ltem	Part No.	Part Name	Qty.
AN	3000-300-114	Cable Tie	3
CA	3001-400-622	NC, TV Label, Inner, Left	1
CA	3001–400–612	NC, TV Label, Inner, Right	1
CC	3001-400-953	Switch Cap	16
CD	3001-400-522	Filler Cap	4
CE	3000–403–831	Speaker With Cable	2

Std. Siderail w/Nurse Call, TV/Radio, G/F, & DMS Auto/Man.

ltem	Part No.	Part Name	Qty.
CB	3001-402-1	DMS Label, Left	1
CB	3001-402-2	DMS Label, Right	1
CC	3001–400–953	Switch Cap	20
CD	3001-400-522	Filler Cap	4
CF	3001-402-30	DMS Module	2
СН	3001–400–633	NC, G/F, DMS Label, Outer, Left	1
СН	3001–400–643	NC, G/F, DMS Label, Outer, Right	1
CJ	3001-402-804	Main to Options PCB Cable, Left	1
CJ	3001-402-803	Main to Options PCB Cable, Right	1


Std. Siderail w/Nurse Call, TV/Radio, G/F, & DMS Auto Only

ltem	Part No.	Part Name	Qty.
CC	3001-400-953	Switch Cap	14
CD	3001-400-522	Filler Cap	6
СН	3001-400-653	NC, G/F, DMS Label, Outer, Left	1
СН	3001-400-663	NC, G/F, DMS Label, Outer, Right	1


Standard Siderail with Nurse Call, TV/Radio, & DMS

ltem	Part No.	Part Name	Qty.
AN	3000-300-114	Cable Tie	3
CA	3001-400-622	NC, TV Label, Inner, Left	1
CA	3001–400–612	NC, TV Label, Inner, Right	1
CC	3001-400-953	Switch Cap	16
CD	3001-400-522	Filler Cap	4
CE	3000-403-831	Speaker With Cable	2

Inner Right Siderail Label

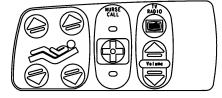

Outer Right Siderail Label

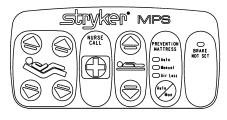





Outer Right Siderail Label






Standard Siderail w/NC, TV/Radio, & DMS Auto/Manual

ltem	Part No.	Part Name	Qty.
СВ	3001-402-1	DMS Label, Left	1
СВ	3001-402-2	DMS Label, Right	1
CC	3001-400-953	Switch Cap	12
CD	3001–400–522	Filler Cap	12
CF	3001-402-30	DMS Module	2
СН	3001–400–635	NC, DMS Label, Outer, Left	1
СН	3001–400–645	NC, DMS Label, Outer, Right	1
CJ	3001-402-804	Main to Options PCB Cable, Left	1
CJ	3001-402-803	Main to Options PCB Cable, Right	1

Standard Siderail w/NC, TV/Radio, & DMS Auto Only


ltem	Part No.	Part Name	Qty.
CC	3001-400-953	Switch Cap	6
CD	3001-400-522	Filler Cap	14
СН	3001–400–655	NC, DMS Label, Outer, Left	1
СН	3001-400-665	NC, DMS Label, Outer, Right	1

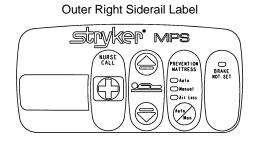


Inner DMS Label




Outer Right Siderail Label

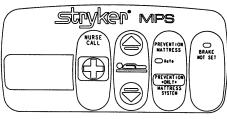


Inner Right Siderail Label

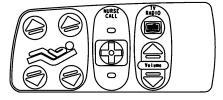


Outer Right Siderail Label

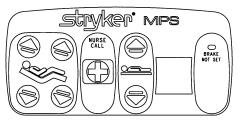

Inner Right Siderail Label

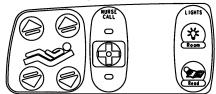

Outer Right Siderail Label

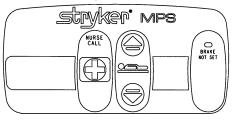

Standard Siderail w/Nurse Call, TV/Radio, & G/F

ltem	Part No.	Part Name	Qty.
CA	3001-400-622	NC, TV Label, Inner, Left	1
CA	3001–400–612	NC, TV Label, Inner, Right	1
CC	3001-400-953	Switch Cap	30
CD	3001-400-522	Filler Cap	10
CE	3000–403–831	Speaker with Cable	2
CF	3001–400–535	Blank Module	2
CH	3001-400-632	NC, G/F Label, Outer, Left	1
CH	3001-400-642	NC, G/F Label, Outer, Right	1

Standard Siderail with Nurse Call and Lights

ltem	Part No.	Part Name	Qty.
CA	3001–400–625	NC, Lights Label, Inner, Left	1
CA	3001–400–615	NC, Lights Label, Inner, Right	1
CC	3001-400-953	Switch Cap	20
CD	3001-400-522	Filler Cap	20
CE	3000–403–831	Speaker With Cable	2
CF	3001-400-535	Blank Module	2
СН	3001–400–631	NC Label, Outer, Left	1
СН	3001–400–641	NC Label, Outer, Right	1

Standard Siderail w/Nurse Call, Lights and DMS

ltem	Part No.	Part Name	Qty.
AN	3000-300-114	Cable Tie	3
CA	3001-400-625	NC, Lights Label, Inner, Left	1
CA	3001–400–615	NC, Lights Label, Inner, Right	1
CC	3001-400-953	Switch Cap	14
CD	3001-400-522	Filler Cap	6
CE	3000–403–831	Speaker With Cable	2

Standard Siderail w/Nurse Call, Lights, & DMS Auto/Man.

ltem	Part No.	Part Name	Qty.
CB	3001-402-1	DMS Label, Left	1
CB	3001-402-2	DMS Label, Right	1
CC	3001-400-953	Switch Cap	12
CD	3001-400-522	Filler Cap	12
CF	3001-402-30	DMS Module	2
СН	3001-400-635	NC, DMS Label, Outer, Left	1
СН	3001–400–645	NC, DMS Label, Outer, Right	1
CJ	3001-402-804	Main to Options PCB Cable, Left	1
CJ	3001-402-803	Main to Options PCB Cable, Right	1

Standard Siderail w/Nurse Call, Lights, & DMS Auto Only

ltem	Part No.	Part Name	Qty.
CC	3001-400-953	Switch Cap	6
CD	3001–400–522	Filler Cap	14
СН	3001-400-655	NC, DMS Label, Outer, Left	1
СН	3001–400–665	NC, DMS Label, Outer, Right	1


Standard Siderail w/Nurse Call, Lights, DMS, & G/F


ltem	Part No.	Part Name	Qty.
AN	3000-300-114	Cable Tie	3
CA	3001-400-625	NC, Lights, DMS Label, Inner, Left	1
CA	3001–400–615	NC, Lights, DMS Label, Inner, Right	1
CC	3001–400–953	Switch Cap	14
CD	3001-400-522	Filler Cap	6
CE	3000-403-831	Speaker with Cable	2

Standard Siderail w/NC, Lights, G/F, & DMS Auto/Man.

ltem	Part No.	Part Name	Qty.
CB	3001-402-1	DMS Label, Left	1
CB	3001-402-2	DMS Label, Right	1
CC	3001–400–953	Switch Cap	20
CD	3001-400-522	Filler Cap	4
CF	3001-402-30	DMS Module	2
CH	3001–400–633	NC, G/F, DMS Label, Outer, Left	1
СН	3001–400–643	NC, G/F, DMS Label, Outer, Right	1
CJ	3001-402-804	Main to Options PCB Cable, Left	1
CJ	3001-402-803	Main to Options PCB Cable, Right	1


Inner Right Siderail Label

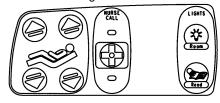


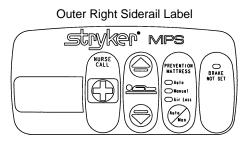



Outer Right Siderail Label

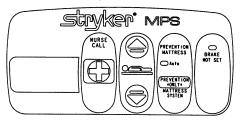

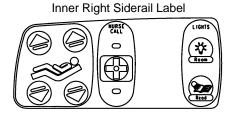







Standard Siderail w/NC, Lights, G/F, & DMS Auto Only

ltem	Part No.	Part Name	Qty.
CC	3001-400-953	Switch Cap	20
CD	3001-400-522	Filler Cap	4
СН	3001-400-653	NC, G/F, DMS Label, Outer, Left	1
СН	3001-400-663	NC, G/F, DMS Label, Outer, Right	1

Standard Siderail with Nurse Call, Lights, and G/F

ltem	Part No.	Part Name	Qty.
CA	3001–400–625	NC, Lights Label Inner, Left	1
CA	3001–400–615	NC, Lights Label, Inner, Right	1
CC	3001-400-953	Switch Cap	28
CD	3001–400–522	Filler Cap	12
CE	3000–403–831	Speaker With Cable	2
CF	3001–400–535	Blank Module	2
CH	3001-400-632	NC, G/F Label, Outer, Left	1
СН	3001-400-642	NC, G/F Label, Outer, Right	1

Standard Siderail with Nurse Call and DMS

ltem	Part No.	Part Name	Qty.
AN	3000-300-114	Cable Tie	3
CA	3001-400-621	Nurse Call Label, Inner, Left	1
CA	3001-400-611	Nurse Call Label, Inner, Right	1
CC	3001-400-953	Switch Cap	10
CD	3001-400-522	Filler Cap	10
CE	3000-403-831	Speaker With Cable	2


Standard Siderail with Nurse Call and DMS Auto/Manual


ltem	Part No.	Part Name	Qty.
СВ	3001-402-1	DMS Label, Left	1
CB	3001-402-2	DMS Label, Right	1
CC	3001-400-953	Switch Cap	12
CD	3001-400-522	Filler Cap	12
CF	3001-402-30	DMS Module	2
CH	3001-400-635	NC, DMS Label, Outer, Left	1
СН	3001-400-645	NC, DMS Label, Outer, Right	1
CJ	3001-402-804	Main to Options PCB Cable, Left	1
CJ	3001-402-803	Main to Options PCB Cable, Right	1


Standard Siderail with Nurse Call and DMS Auto Only

ltem	Part No.	Part Name	Qty.
CC	3001-400-953	Switch Cap	6
CD	3001-400-522	Filler Cap	14
СН	3001-400-655	NC, DMS Label, Outer, Left	1
СН	3001–400–665	NC, DMS Label, Outer, Right	1


Outer Right Siderail Label

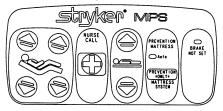

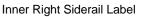


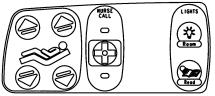

Outer Right Siderail Label

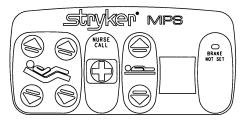


Inner Right Siderail Label CALL 0

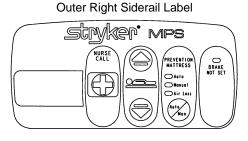





Outer Right Siderail Label

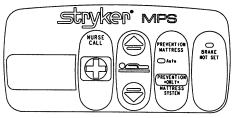

Label

Standard Siderail with Nurse Call, DMS, & G/F

ltem	Part No.	Part Name	Qty.
AN	3000-300-114	Cable Tie	3
CA	3001–400–621	Nurse Call Label, Inner, Left	1
CA	3001–400–611	Nurse Call Label, Inner, Right	1
CC	3001-400-953	Switch Cap	10
CD	3001-400-522	Filler Cap	10
CE	3000–403–831	Speaker with Cable	2

Standard Siderail with Nurse Call, G/F, & DMS Auto/Manual

ltem	Part No.	Part Name	Qty.
CB	3001-402-1	DMS Label, Left	1
CB	3001-402-2	DMS Label, Right	1
CC	3001-400-953	Switch Cap	20
CD	3001-400-522	Filler Cap	4
CF	3001-402-30	DMS Module	2
СН	3001–400–633	NC, G/F, DMS Label, Outer, Left	1
СН	3001–400–643	NC, G/F, DMS Label, Outer, Right	1
CJ	3001-402-804	Main to Options PCB Cable, Left	1
CJ	3001-402-803	Main to Options PCB Cable, Right	1


Standard Siderail with Nurse Call, G/F, & DMS Auto Only

ltem	Part No.	Part Name	Qty.
CC	3001-400-953	Switch Cap	14
CD	3001-400-522	Filler Cap	6
СН	3001-400-653	NC, G/F, DMS Label, Outer, Left	1
СН	3001-400-663	NC, G/F, DMS Label, Outer, Right	1

Standard Siderail with Nurse Call and G/F

ltem	Part No.	Part Name	Qty.
CA	3001–400–621	Nurse Call Label, Inner, Left	1
CA	3001–400–611	Nurse Call Label, Inner, Right	1
CC	3001-400-953	Switch Cap	24
CD	3001-400-522	Filler Cap	16
CE	3000–403–831	Speaker With Cable	2
CF	3001-400-535	Blank Module	2
СН	3001–400–632	NC, G/F Label, Outer, Left	1
СН	3001-400-642	NC, G/F Label, Outer, Right	1

Inner Right Siderail Label

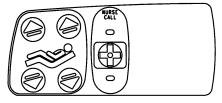


Inner DMS


Outer Right Siderail Label






Outer Right Siderail Label

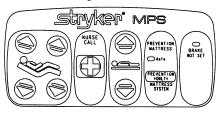

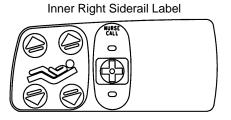

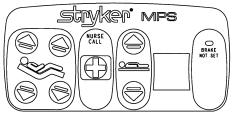


Standard Siderail with TV/Radio

ltem	Part No.	Part Name	Qty.
CA	3001-400-626	TV Label, Inner, Left	1
CA	3001–400–616	TV Label, Inner, Right	1
CC	3001–400–953	Switch Cap	18
CD	3001-400-522	Filler Cap	22
CE	3000–403–831	Speaker With Cable	2
CF	3001–400–535	Blank Module	2
СН	3001–400–630	NC, G/F Label, Outer, Left	1
СН	3001–400–640	NC, G/F Label, Outer, Right	1

Inner Right Siderail Label




Outer Right Siderail Label

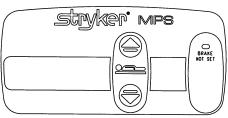


Inner Right Siderail Label

Standard Siderail with TV/Radio and Lights


ltem	Part No.	Part Name	Qty.
CA	3001–400–624	TV, Lights Label, Inner, Left	1
CA	3001–400–614	TV, Lights Label, Inner, Right	1
CC	3001–400–953	Switch Cap	26
CD	3001-400-522	Filler Cap	14
CE	3000–403–831	Speaker with Cable	2
CF	3001-400-535	Blank Module	2
СН	3001–400–630	Standard Label, Outer, Left	1
СН	3001-400-640	Standard Label, Outer, Right	1

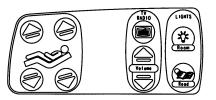


Outer Right Siderail Label

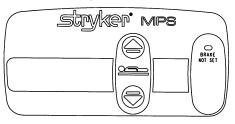

Inner Right Siderail Label

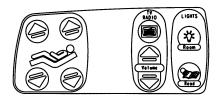

Outer Right Siderail Label

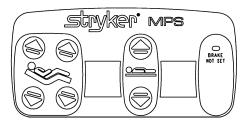



Standard Siderail with TV/Radio, Lights, and G/F

ltem	Part No.	Part Name	Qty.
CA	3001-400-624	TV, Lights Label, Inner, Left	1
CA	3001-400-614	TV, Lights Label, Inner, Right	1
CC	3001-400-953	Switch Cap	30
CD	3001-400-522	Filler Cap	10
CE	3000-403-831	Speaker with Cable	2
CF	3001-400-535	Blank Module	2
СН	3001-400-636	G/F Label, Outer, Left	1
СН	3001-400-646	G/F Label, Outer, Right	1

Standard Siderail with TV/Radio, Lights, & DMS

ltem	Part No.	Part Name	Qty.
AN	3000-300-114	Cable Tie	3
CA	3001-400-624	TV, Lights Label, Inner, Left	1
CA	3001–400–614	TV, Lights Label, Inner, Right	1
CC	3001-400-953	Switch Cap	18
CD	3001-400-522	Filler Cap	2
CE	3000–403–831	Speaker With Cable	2

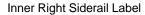

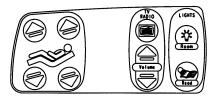

Standard Siderail with TV/Radio, Lights, & DMS Auto/Manual

ltem	Part No.	Part Name	Qty.
CB	3001-402-1	DMS Label, Left	1
CB	3001-402-2	DMS Label, Right	1
CC	3001-400-953	Switch Cap	10
CD	3001-400-522	Filler Cap	14
CF	3001-402-30	DMS Module	2
СН	3001-400-637	DMS Label, Outer, Left	1
СН	3001–400–647	DMS Label, Outer, Right	1
CJ	3001-402-804	Main to Options PCB Cable, Left	1
CJ	3001-402-803	Main to Options PCB Cable, Right	1


Standard Siderail with TV/Radio, Lights, & DMS Auto Only

ltem	Part No.	Part Name	Qty.
CC	3001-400-953	Switch Cap	4
CD	3001-400-522	Filler Cap	16
СН	3001-400-657	DMS Label, Outer, Left	1
СН	3001–400–667	DMS Label, Outer, Right	1

Outer Right Siderail Label




Outer Right Siderail Label

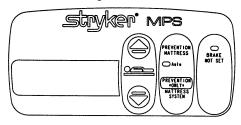

Standard Siderail with TV/Radio, Lights, G/F, and DMS

ltem	Part No.	Part Name	Qty.
AN	3000-300-114	Cable Tie	3
CA	3001–400–624	TV, Lights Label, Inner, Left	1
CA	3001–400–614	TV, Lights Label, Inner, Right	1
CC	3001–400–953	Switch Cap	18
CD	3001-400-522	Filler Cap	2
CE	3000–403–831	Speaker With Cable	2

Inner Right Siderail Label

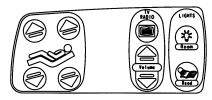

Std. Siderail with TV/Radio, Lights, G/F, & DMS Auto/Man.

ltem	Part No.	Part Name	Qty.
СВ	3001-402-1	DMS Label, Left	1
СВ	3001-402-2	DMS Label, Right	1
CC	3001–400–953	Switch Cap	18
CD	3001–400–522	Filler Cap	6
CF	3001-402-30	DMS Module	2
CH	3001–400–634	DMS, G/F Label, Outer, Left	1
CH	3001–400–644	DMS, G/F Label, Outer, Right	1
CJ	3001-402-804	Main to Options PCB Cable, Left	1
CJ	3001-402-803	Main to Options PCB Cable, Right	1

Std. Siderail with TV/Radio, Lights, G/F, & DMS Auto Only

ltem	Part No.	Part Name	Qty.
CC	3001-400-953	Switch Cap	12
CD	3001-400-522	Filler Cap	8
СН	3001-400-654	DMS, G/F Label, Outer, Left	1
СН	3001-400-664	DMS, G/F Label, Outer, Right	1

Standard Siderail with TV/Radio, G/F and DMS


ltem	Part No.	Part Name	Qty.
AN	3000-300-114	Cable Tie	3
CA	3001-400-626	TV Label, Inner, Left	1
CA	3001-400-616	TV Label, Inner, Right	1
CC	3001-400-953	Switch Cap	14
CD	3001-400-522	Filler Cap	6
CE	3000-403-831	Speaker With Cable	2

Standard Siderail with TV/Radio, G/F and DMS Auto/Man.


ltem	Part No.	Part Name	Qty.
СВ	3001-402-1	DMS Label, Left	1
СВ	3001-402-2	DMS Label, Right	1
CC	3001-400-953	Switch Cap	18
CD	3001-400-522	Filler Cap	6
CF	3001-402-30	DMS Module	2
CH	3001–400–634	DMS, G/F Label, Outer, Left	1
CH	3001–400–644	DMS, G/F Label, Outer, Right	1
CJ	3001-402-804	Main to Options PCB Cable, Left	1
CJ	3001–402–803	Main to Options PCB Cable, Right	1


Standard Siderail with TV/Radio, G/F and DMS Auto Only


ltem	Part No.	Part Name	Qty.
CC	3001-400-953	Switch Cap	12
CD	3001-400-522	Filler Cap	8
СН	3001–400–654	DMS, G/F Label, Outer, Left	1
СН	3001-400-664	DMS, G/F Label, Outer, Right	1

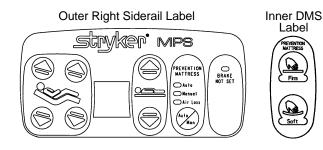




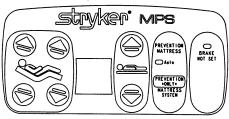

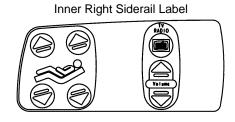

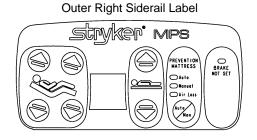



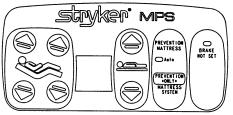

Standard Siderail with TV/Radio and DMS

ltem	Part No.	Part Name	Qty.
AN	3000-300-114	Cable Tie	3
CA	3001–400–626	TV Label, Inner, Left	1
CA	3001–400–616	TV Label, Inner, Right	1
CC	3001–400–953	Switch Cap	14
CD	3001-400-522	Filler Cap	6
CE	3000–403–831	Speaker With Cable	2

Standard Siderail with TV/Radio and DMS Auto/Manual

ltem	Part No.	Part Name	Qty.
CB	3001-402-1	DMS Label, Left	1
CB	3001-402-2	DMS Label, Right	1
CC	3001-400-953	Switch Cap	10
CD	3001-400-522	Filler Cap	14
CF	3001-402-30	DMS Module	2
СН	3001–400–637	DMS Label, Outer, Left	1
СН	3001–400–647	DMS Label, Outer, Right	1
CJ	3001-402-804	Main to Options PCB Cable, Left	1
CJ	3001-402-803	Main to Options PCB Cable, Right	1


Standard Siderail with TV/Radio and DMS Auto Only


ltem	Part No.	Part Name	Qty.
CC	3001-400-953	Switch Cap	4
CD	3001-400-522	Filler Cap	16
СН	3001-400-657	DMS Label, Outer, Left	1
СН	3001-400-667	DMS Label, Outer, Right	1

Standard Siderail with TV/Radio and G/F

ltem	Part No.	Part Name	Qty.
CA	3001–400–626	TV Label, Inner, Left	1
CA	3001–400–616	TV Label, Inner, Right	1
CC	3001–400–953	Switch Cap	26
CD	3001-400-522	Filler Cap	14
CE	3000-403-831	Speaker with Cable	2
CF	3001–400–535	Blank Module	2
СН	3001-400-636	G/F Label, Outer, Left	1
СН	3001-400-646	G/F Label, Outer, Right	1


Inner Right Siderail Label

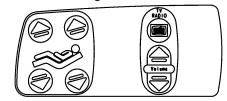

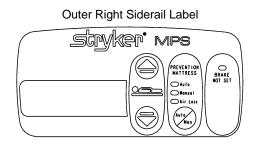





Outer Right Siderail Label

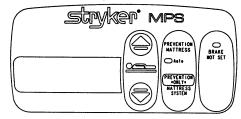

Inner Right Siderail Label

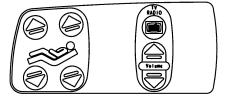

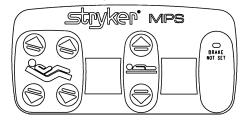

Standard Siderail with Lights and DMS

ltem	Part No.	Part Name	Qty.
AN	3000-300-114	Cable Tie	3
CA	3001-400-627	Lights Label, Inner, Left	1
CA	3001–400–617	Lights Label, Inner, Right	1
CC	3001-400-953	Switch Cap	12
CD	3001-400-522	Filler Cap	8
CE	3001–400–517	Speaker Seal	2

Standard Siderail with Lights and DMS Auto/Manual

ltem	Part No.	Part Name	Qty.
СВ	3001-402-1	DMS Label, Left	1
СВ	3001-402-2	DMS Label, Right	1
CC	3001-400-953	Switch Cap	10
CD	3001-400-522	Filler Cap	14
CF	3001-402-30	DMS Module	2
СН	3001–400–637	DMS Label, Outer, Left	1
СН	3001-400-647	DMS Label, Outer, Right	1
CJ	3001-402-804	Main to Options PCB Cable, Left	1
CJ	3001-402-803	Main to Options PCB Cable, Right	1

Standard Siderail with Lights and DMS Auto Only

ltem	Part No.	Part Name	Qty.
CC	3001-400-953	Switch Cap	4
CD	3001-400-522	Filler Cap	16
СН	3001-400-657	DMS Label, Outer, Left	1
СН	3001-400-667	DMS Label, Outer, Right	1


Standard Siderail with Lights, G/F and DMS


ltem	Part No.	Part Name	Qty.
AN	3000-300-114	Cable Tie	3
CA	3001–400–627	Lights Label, Inner, Left	1
CA	3001–400–617	Lights Label, Inner, Right	1
CC	3001–400–953	Switch Cap	12
CD	3001–400–522	Filler Cap	18
CE	3001–400–517	Speaker Seal	2

Standard Siderail with Lights, G/F and DMS Auto/Manual

ltem	Part No.	Part Name	Qty.
СВ	3001-402-1	DMS Label, Left	1
СВ	3001-402-2	DMS Label, Right	1
CC	3001-400-953	Switch Cap	18
CD	3001-400-522	Filler Cap	6
CF	3001-402-30	DMS Module	2
СН	3001–400–634	DMS, G/F Label, Outer, Left	1
СН	3001–400–644	DMS, G/F Label, Outer, Right	1
CJ	3001-402-804	Main to Options PCB Cable, Left	1
CJ	3001-402-803	Main to Options PCB Cable, Right	1


Inner Right Siderail Label

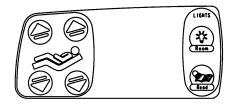

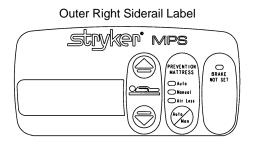




Outer Right Siderail Label

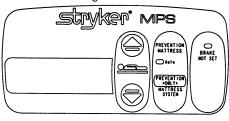

Inner Right Siderail Label

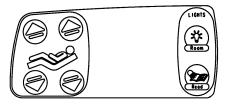






Standard Siderail with Lights, G/F and DMS Auto Only

ltem	Part No.	Part Name	Qty.
CC	3001–400–953	Switch Cap	12
CD	3001-400-522	Filler Cap	8
СН	3001-400-654	DMS, G/F Label, Outer, Left	1
СН	3001-400-664	DMS, G/F Label, Outer, Right	1


Outer Right Siderail Label

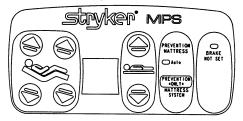


Standard Siderail with Lights and Gatch/Fowler

ltem	Part No.	Part Name	Qty.
CA	3001-400-627	Lights Label, Inner, Left	1
CA	3001–400–617	Lights Label, Inner, Right	1
CC	3001-400-953	Switch Cap	24
CD	3001-400-522	Filler Cap	16
CE	3001–400–517	Speaker Seal	2
CF	3001–400–535	Blank Module	2
CH	3001-400-636	G/F Label, Outer, Left	1
CH	3001–400–646	G/F Label, Outer, Right	1

Inner Right Siderail Label

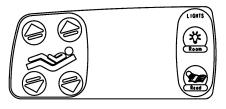


Outer Right Siderail Label

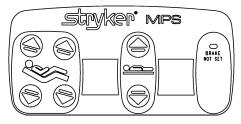


Standard Siderail with Lights

ltem	Part No.	Part Name	Qty.
CA	3001–400–627	Lights Label, Inner, Left	1
CA	3001–400–617	Lights Label, Inner, Right	1
CC	3001-400-953	Switch Cap	16
CD	3001–400–522	Filler Cap	24
CE	3001–400–517	Speaker Seal	2
CF	3001–400–535	Blank Module	2
CH	3001-400-630	Standard Label, Outer, Left	1
СН	3001-400-640	Standard Label, Outer, Right	1

Inner Right Siderail Label

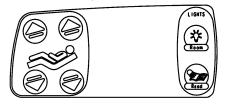

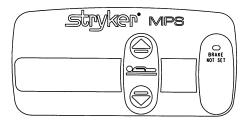


1

1


Standard Siderail with Gatch/Fowler

ltem	Part No.	Part Name
CA	3001-400-620	Standard Label Inner, Left
CA	3001-400-610	Standard Label, Inner, Right
CC	3001-400-953	Switch Cap
CD	3001-400-522	Filler Cap
CE	3001-400-517	Speaker Seal
CF	3001–400–535	Blank Module
СН	3001-400-636	G/F Label, Outer, Left
СН	3001-400-646	G/F Label, Outer, Right

Inner Right Siderail Label

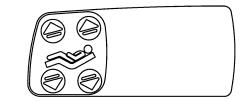

Outer Right Siderail Label

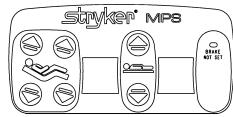

Standard Siderail with DMS

ltem	Part No.	Part Name	Qty.
AN	3000-300-114	Cable Tie	3
CA	3001-400-620	Standard Label, Inner, Left	1
CA	3001-400-610	Standard Label, Inner, Right	1
CC	3001-400-953	Switch Cap	8
CD	3001-400-522	Filler Cap	12
CE	3001–400–517	Speaker Seal	2


Standard Siderail with DMS Auto/Manual

ltem	Part No.	Part Name	Qty.
СВ	3001-402-1	DMS Label, Left	1
СВ	3001-402-2	DMS Label, Right	1
CC	3001-400-953	Switch Cap	10
CD	3001-400-522	Filler Cap	14
CF	3001-402-30	DMS Module	2
CH	3001-400-637	DMS Label, Outer, Left	1
CH	3001-400-647	DMS Label, Outer, Right	1
CJ	3001-402-804	Main to Options PCB Cable, Left	1
CJ	3001-402-803	Main to Options PCB Cable, Right	1

Standard Siderail with DMS Auto Only


ltem	Part No.	Part Name	Qty.
CC	3001-400-953	Switch Cap	4
CD	3001-400-522	Filler Cap	16
СН	3001-400-657	DMS Label, Outer, Left	1
СН	3001–400–667	DMS Label, Outer, Right	1

Inner Right Siderail Label

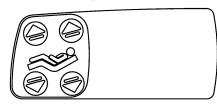




Outer Right Siderail Label

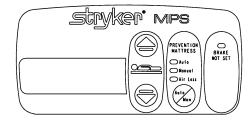

Outer Right Siderail Label

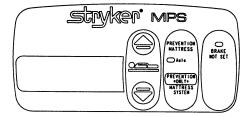

Standard Siderail with DMS & G/F

ltem	Part No.	Part Name	Qty.
AN	3000-300-114	Cable Tie	3
CA	3001-400-620	Standard Label, Inner, Left	1
CA	3001–400–610	Standard Label, Inner, Right	1
CC	3001–400–953	Switch Cap	8
CD	3001-400-522	Filler Cap	12
CE	3001–400–517	Speaker Seal	2


Standard Siderail with G/F and DMS Auto/Manual


ltem	Part No.	Part Name	Qty.
CB	3001-402-1	DMS Label, Left	1
CB	3001-402-2	DMS Label, Right	1
CC	3001-400-953	Switch Cap	18
CD	3001-400-522	Filler Cap	6
CF	3001-402-30	DMS Module	2
СН	3001–400–634	DMS, G/F Label, Outer, Left	1
СН	3001–400–644	DMS, G/F Label, Outer, Right	1
CJ	3001-402-804	Main to Options PCB Cable, Left	1
CJ	3001-402-803	Main to Options PCB Cable, Right	1

Standard Siderail with G/F and DMS Auto Only

ltem	Part No.	Part Name	Qty.
CC	3001-400-953	Switch Cap	12
CD	3001-400-522	Filler Cap	8
СН	3001-400-654	DMS, G/F Label, Outer, Left	1
СН	3001-400-664	DMS, G/F Label, Outer, Right	1


Inner Right Siderail Label

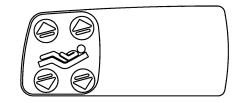


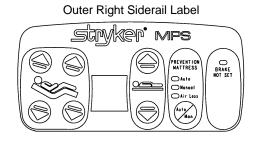




Outer Right Siderail Label

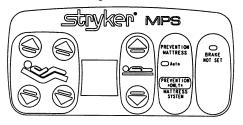

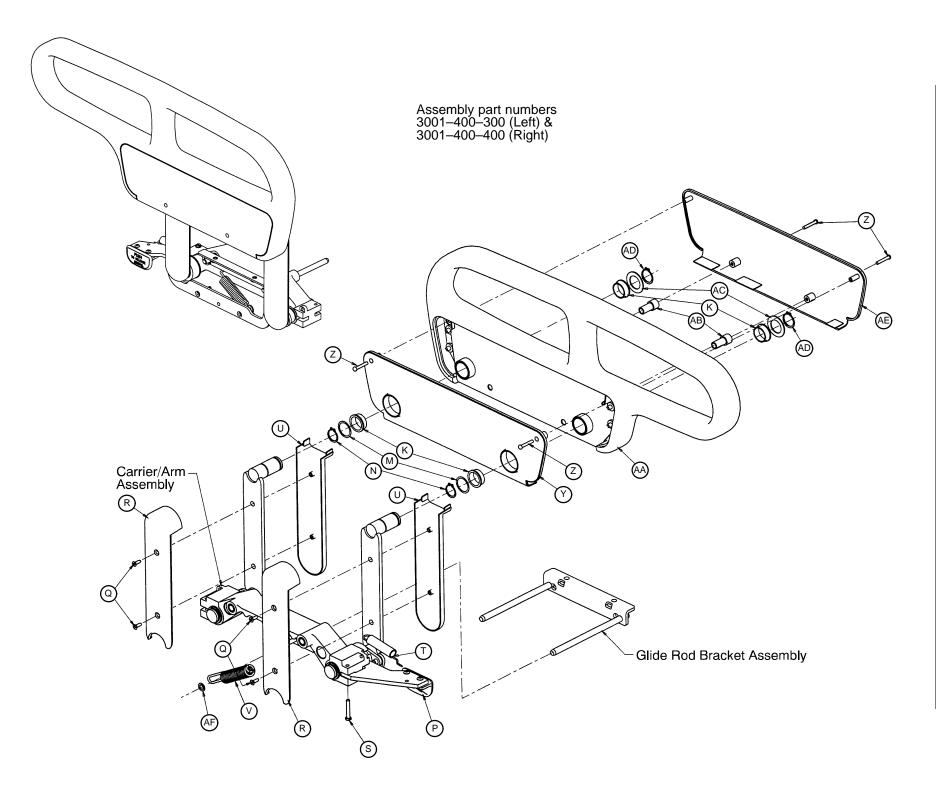

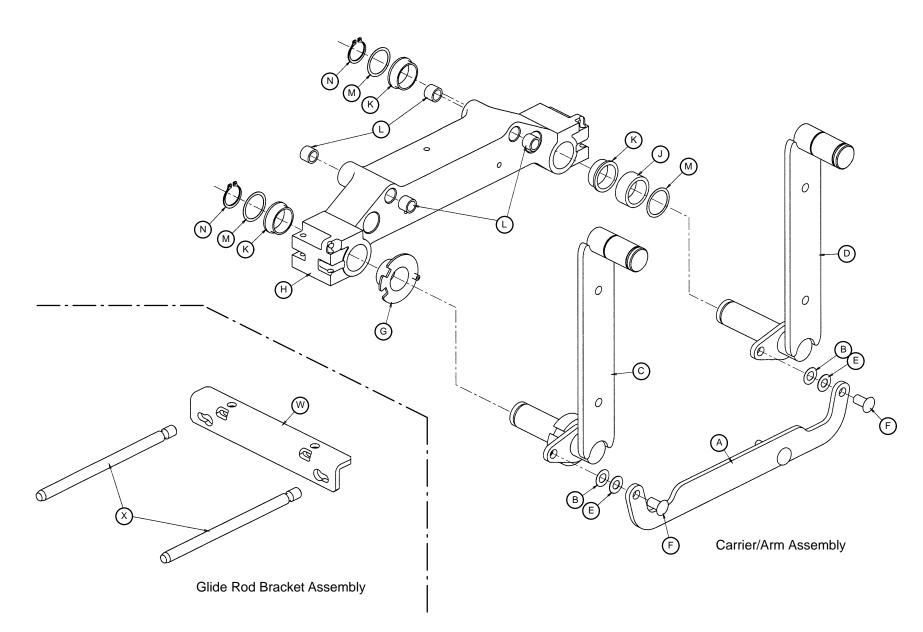

ltem	Part No.	Part Name	Qty.	ltem	Part No.	Part Name	Qty.
А	3001-400-325	Timing Link Weldment, Lt.	1	А	3001-400-425	Timing Link Weldment, Rt.	1
В	11–403	Shim Washer	2	В	11–403	Shim Washer	2
D	3001-400-327	Foot Arm Wldmt., Lt., Head	1 1	D	3001-400-427	Foot Arm Wldmt., Rt., Hea	d 1
Е	11–377	Nylon Washer	2	Е	11–377	Nylon Washer	2
F	3001-400-501	Siderail Linkage Rivet	2	F	3001–400–501	Siderail Linkage Rivet	2
G	3001-400-513	Wear Bushing	1	G	3001–400–513	Wear Bushing	1
Н	3001-400-528	Siderail Carrier	1	Н	3001–400–528	Siderail Carrier	1
J	3001-400-530	Siderail Arm Spacer	1	J	3001–400–530	Siderail Arm Spacer	1
K	3000-400-513	Flange Bearing	7	K	3000–400–513	Flange Bearing	7
L	3000-400-557	Sleeve Bearing	4	I	3000–400–557	Sleeve Bearing	4
М	11–353	Shim Washer	5	М	11–353	Shim Washer	5
Ν	28–128	Retaining Ring	4	Ν	28–128	Retaining Ring	4
Р	(page 185)	Release Lever, Left	1	Р	(page 186)	Release Lever, Right	1
Q	23–261	Self–Tapping Screw	4	Q	23–261	Self–Tapping Screw	4
R	5000–20–5	Inner Arm Cover	2	R	5000–20–5	Inner Arm Cover	2
S	3–330	Hex Washer Hd. Screw	1	S	3–330	Hex Washer Hd. Screw	1
Т	3000-200-334	Extension Spring	1	Т	3000–200–334	Extension Spring	1
U	5000–20–6	Outer Arm Cover	2	U	5000-20-6	Outer Arm Cover	2
V	3000-400-545	Tension Spring	1	V	3000–400–545	Tension Spring	1
W	3001–400–553	Mounting Bracket	1	W	3001–400–553	Mounting Bracket	1
Х	3001-400-554	Glide Rod	2	Х	3001–400–554	Glide Rod	2
Y	3001-400-526	Inner Panel	1	Y	3001–400–526	Inner Panel	1
Z	23–90	High–Low Tapping Screw	4	Z	23–90	High–Low Tapping Screw	4
AA	(page 183)	Support, Left	1	AA	(page 184)	Support, Right	1
AB	3000-400-523	Panel Spacer	2	AB	3000–400–523	Panel Spacer	2
AC	11–343	Shim Washer	2	AC	11–343	Shim Washer	2
AD	28–132	Bowed Retaining Ring	2	AD	28–132	Bowed Retaining RIng	2
AE	3001-400-527	Outer Panel	1	AE	3001–400–527	Outer Panel	1
AF	28–171	Push–On Retaining Ring	1	AF	28–171	Push–On Retaining Ring	1

3001–400–300 Left Standard Components

3001–400–400 Right Standard Components

3001-442-300 Left	Rail – 8"	Shorter Bed
-------------------	-----------	-------------

ltem	Part No.	Part Name	Qty.
С	3001-442-318	Foot Arm Wldmt., Lt., Foot	1

3001-442-400 Right Rail - 8" Shorter Bed

ltem	Part No.	Part Name	Qty.
С	3001-442-418	Foot Arm Wldmt., Rt., Foot	1

3001-443-300 L	Left Rail –	Standard	Bed
----------------	-------------	----------	-----


Item Part No.

С


3001-443-400 Right Rail - Standard Bed

Part No.	Part Name	Qty.	ltem	Part No.	Part Name	Qty.
3001-400-328	Foot Arm Wldmt., Lt., Foot	1	С	3001-400-428	Foot Arm Wldmt., Rt., Foo	ot 1


Assembly part number 3001-400-360

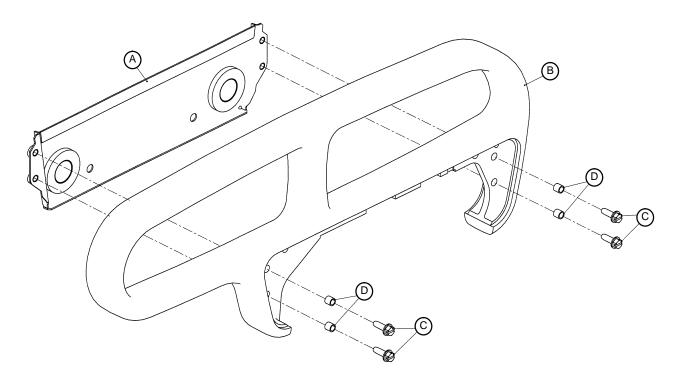


Item	Part No.	Part Name	Qty.
А	3001-400-31	Foot Support Weldment	1
В	3000-400-520	Foot Rail	1
С	3–224	Hex Washer Hd. Screw	6
D	3000-400-558	Siderail Spacer	6

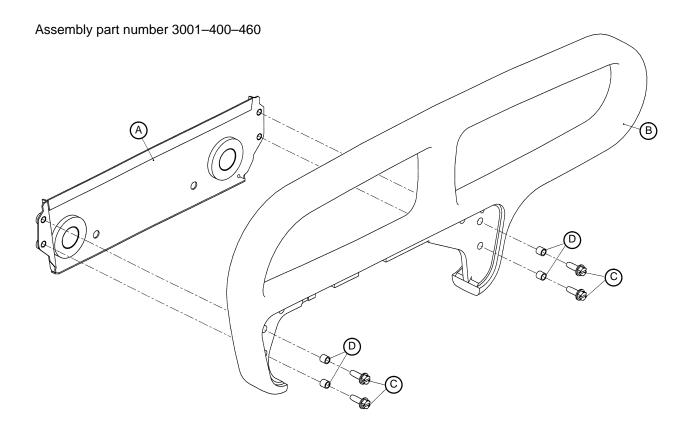


ltem	Part No.	Part Name	Qty.
А	3001–400–580	Foot Support Weldment	1
В	3000-400-520	Foot Rail	1
С	3–226	Hex Washer Hd. Screw	4
D	3001–400–558	Siderail Spacer	4

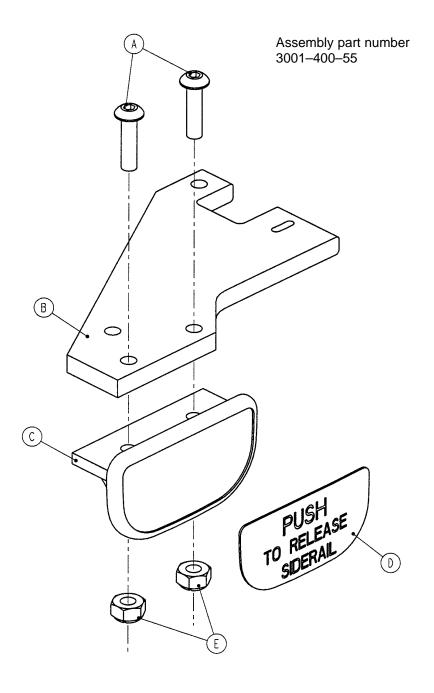

Item	Part No.	Part Name	Qty.
А	4–278	Button Hd. Screw	2
В	3001-400-552	Release Lever	1
С	3001-400-514	Release Lever Pad	1
D	3001–400–505	Pad Label	1
E	16–2	Nylock Nut	2

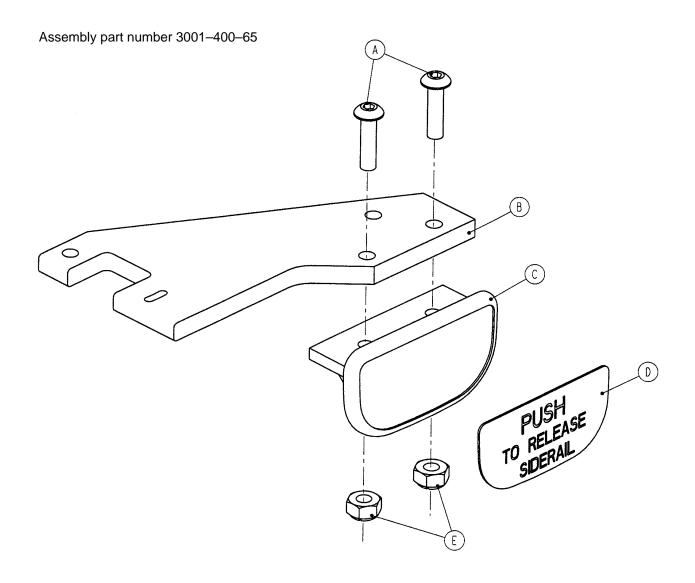

ltem	Part No.	Part Name	Qty.
А	4–278	Button Hd. Screw	2
В	3001-400-552	Release Lever	1
С	3001–400–514	Release Lever Pad	1
D	3001-400-505	Pad Label	1
E	16–2	Nylock Nut	2

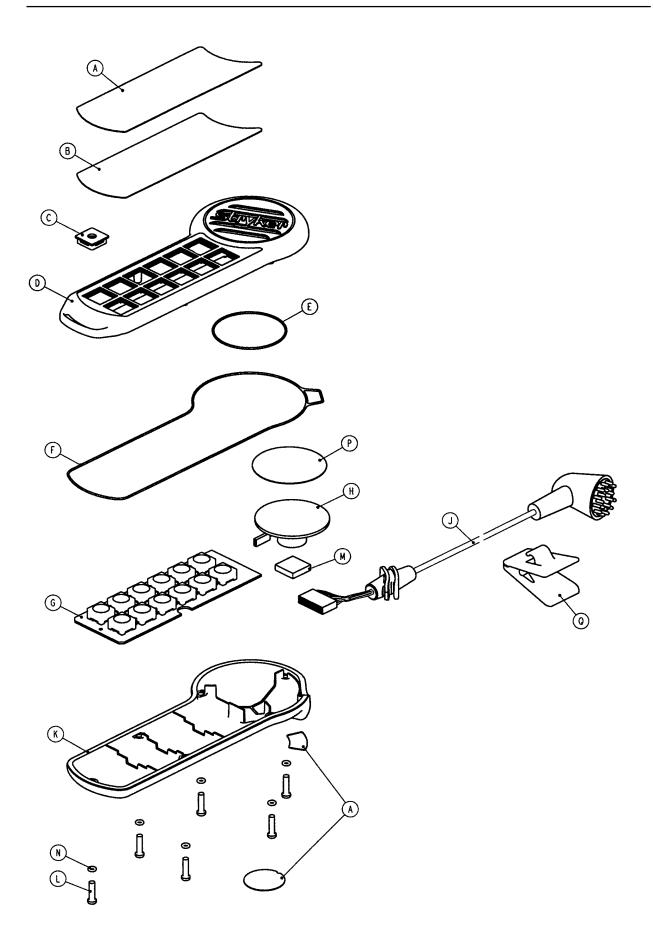

3001–315–11 Combination Pendant – Motion/Comm./DMS

ltem	Part No.	Part Name	Qty.
А	3001–315–41	Mot./Comm./DMS Label	1
В	3001–315–51	Mot./Comm./DMS Seal	1
D	3001–315–30	Upper Housing	1
Е	45–984	Speaker O–Ring	1
F	3001–315–36	Housing O–Ring	1
G	3001–315–900	PCB Assembly	1
Н	3001–315–801	Speaker Cable	1
J	3001–315–800	Pendant Cable	1
K	3001–315–31	Lower Housing	1
L	23–260	Tapping Screw	6
Μ	3001–315–37	Speaker Foam	1
Ν	45–6	Screw O–Ring	6
Q	52-807	Cord Clamp	1

3001–315–12 Combination Pendant – Motion/Communication

ltem	Part No.	Part Name	Qty.
А	3001-315-42	Motion/Comm. Label	1
В	3001–315–52	Motion/Comm. Seal	1
С	3001-400-522	Filler Cap	2
D	3001–315–30	Upper Housing	1
E	45–984	Speaker O–Ring	1
F	3001-315-36	Housing O–Ring	1
G	3001–315–900	PCB Assembly	1
Н	3001–315–801	Speaker Cable	1
J	3001–315–800	Pendant Cable	1
K	3001-315-31	Lower Housing	1
L	23–260	Tapping Screw	6
Μ	3001–315–37	Speaker Foam	1
Р	45–6	Screw O–Ring	6
S	52-807	Cord Clamp	1

3001–315–13 Combination Pendant – Motion/DMS



ltem	Part No.	Part Name	Qty.
А	3001–315–43	Motion/DMS Label	1
В	3001–315–53	Motion/DMS Seal	1
С	3001-400-522	Filler Cap	6
D	3001-315-30	Upper Housing	1
F	3001–315–36	Housing O–Ring	1
G	3001–315–910	PCB Assembly	1
J	3001–315–800	Pendant Cable	1
K	3001–315–31	Lower Housing	1
L	23–260	Tapping Screw	6
Ν	45–6	Screw O–Ring	6
Р	3001–315–71	Speaker Blank	1
Q	52-807	Cord Clamp	1

3001–315–14 Combination Pendant – Motion Only



ltem	Part No.	Part Name	Qty.
А	3001–315–44	Motion Label	1
В	3001–315–54	Motion Seal	1
С	3001-400-522	Filler Cap	8
D	3001–315–30	Upper Housing	1
F	3001–315–36	Housing O–Ring	1
G	3001–315–910	PCB Assembly	1
J	3001–315–800	Pendant Cable	1
K	3001–315–31	Lower Housing	1
L	23–260	Tapping Screw	6
Ν	45–6	Screw O–Ring	6
Р	3001–315–71	Speaker Blank	1
Q	52-807	Cord Clamp	1

3001–315–15 Combination Pendant – Communication/DMS

ltem	Part No.	Part Name	Qty.
А	3001–315–45	Comm./DMS Label	1
В	3001–315–55	Comm./DMS Seal	1
С	3001-400-522	Filler Cap	4
D	3001–315–30	Upper Housing	1
E	45–984	Speaker O–Ring	1
F	3001–315–36	Housing O–Ring	1
G	3001–315–900	PCB Assembly	1
Н	3001–315–801	Speaker Cable	1
J	3001–315–800	Pendant Cable	1
K	3001–315–31	Lower Housing	1
L	23–260	Tapping Screw	6
Μ	3001–315–37	Speaker Foam	1
N	45–6	Screw O–Ring	6
Q	52-807	Cord Clamp	1

3001–315–16 Combination Pendant – Communication Only

ltem	Part No.	Part Name	Qty.
А	3001–315–46	Communication Label	1
В	3001–315–56	Communication Seal	1
С	3001-400-522	Filler Cap	6
D	3001–315–30	Upper Housing	1
Е	45–984	Speaker O-Ring	1
F	3001–315–36	Housing O–Ring	1
G	3001–315–900	PCB Assembly	1
Н	3001–315–801	Speaker Cable	1
J	3001–315–800	Pendant Cable	1
K	3001–315–31	Lower Housing	1
L	23–260	Tapping Screw	6
М	3001–315–37	Speaker Foam	1
Ν	45–6	Screw O–Ring	6
Q	52-807	Cord Clamp	1









ltem	Part No.	Part Name	Qty.
А	3001–315–47	Motion/DMS/NC Label	1
В	3001–315–57	Motion/DMS/NC Seal	1
С	3001-400-522	Filler Cap	5
D	3001–315–30	Upper Housing	1
F	3001–315–36	Housing O–Ring	1
G	3001–315–910	PCB Assembly	1
J	3001–315–800	Pendant Cable	1
K	3001–315–31	Lower Housing	1
L	23–260	Tapping Screw	6
Ν	45–6	Screw O–Ring	6
Р	3001–315–71	Speaker Blank	1
Q	52-807	Cord Clamp	1


3001–315–17 Combination Pendant – Motion/DMS/NC


3001–315–18 Combination Pendant – Motion/NC

0	ltem	Part No.	Part Name	Qty.
	А	3001–315–48	Motion/NC Label	1
	В	3001–315–58	Motion/NC Seal	1
	С	3001-400-522	Filler Cap	7
	D	3001–315–30	Upper Housing	1
	F	3001–315–36	Housing O–Ring	1
KNEE	G	3001–315–910	PCB Assembly	1
\leq	J	3001–315–800	Pendant Cable	1
	K	3001–315–31	Lower Housing	1
	L	23–260	Tapping Screw	6
	Ν	45–6	Screw O–Ring	6
	Р	3001–315–71	Speaker Blank	1
	Q	52-807	Cord Clamp	1

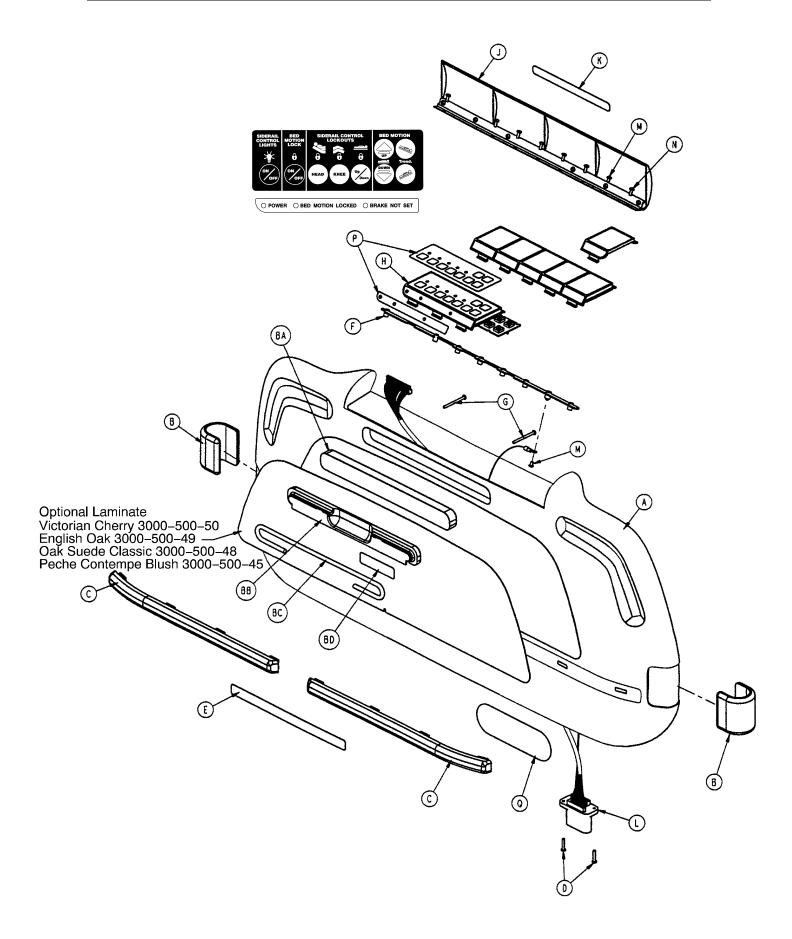

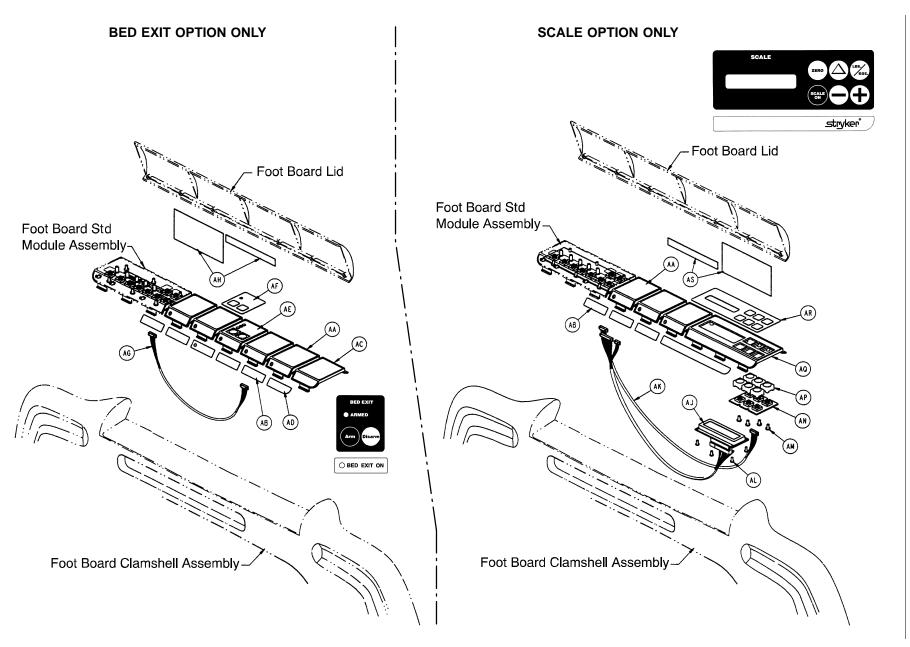

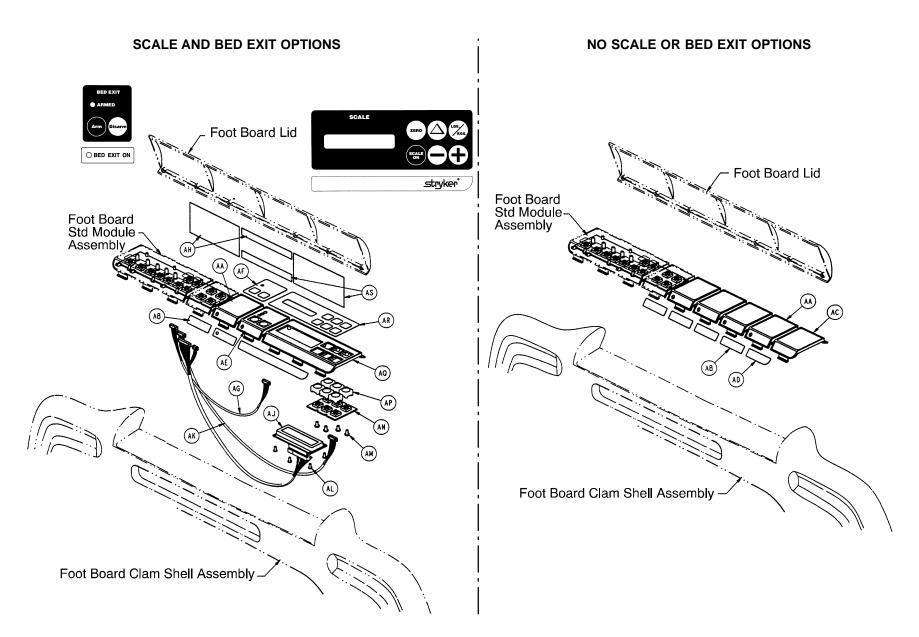

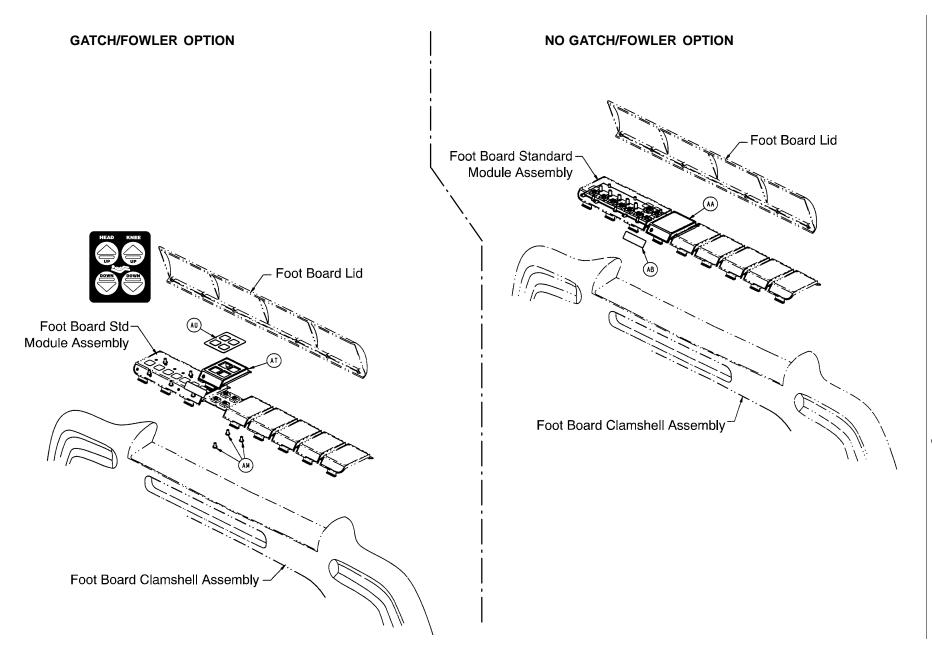

Foot Board Assembly

3001-500-11 Standard Components

ltem	Part No.	Part Name	Qty.	ltem	Part No.	Part Name	Qty.
A B C D E F G	3001-500-10 3000-500-7 3000-500-6 50-39 3000-500-29 3001-500-64 23-99	Clam Shell Assembly "C" Bumper Bumper Strip Pan Hd. Mach. Screw Hazard Label Hinge Plate Phillips Pan Hd. Screw	1 2 2 1 1 2	HJKLMNP	3001–500–28 3001–500–1 3000–500–25 3001–500–801 50–38 23–103 3001–500–24	Std. Control Module Ass'y Lid Assembly Lid Label Foot Board Drawer Cable Pan Hd. Mach. Screw Pan Hd. Hi–Lo Tap. Screw Ft. Bd. Std. Module Label	1 1 1 2 7 1
				Q	3001–500–601	"Secure" Logo Label	1

Item Part No.

ΒA

3000-525-000 Chart/Constavac Hanger Option

ltem	Part No.	Part Name	Qty.
BB	3000-525-1	Chart Rack	1
BC	3000-525-2	Chart Rod	1
BD	3000–525–4	Chart Rack Label	1

3001–508–11 Bed Exit Option Only

ltem	Part No.	Part Name	Qty.
AA	3001–500–3	Blank Module	4
AB	3000-500-26	Blank Module Label	4
AC	3000-500-4	End Module	1
AD	3000-500-27	End Module Label	1
AE	3001–508–30	Bed Exit Module Assembly	1
AF	3001–508–2	Bed Exit Module Label	1
AG	3001-508-800	Bed Exit Keypad Cable	1
AH	3000–508–11	Bed Exit Label	1

3001-507-21 Scale and Bed Exit Options

ltem	Part No.	Part Name	Qty.
AA	3001–500–3	Blank Module	1
AB	3000-500-26	Blank Module Label	1
AE	3001–508–30	Bed Exit Module Ass'y	1
AF	3001–508–2	Bed Exit Module Label	1
AG	3001-508-800	Bed Exit Keypad Cable	1
AH	3000–508–11	Bed Exit Label	1
AJ	3001-507-900	Scale LCD Display Cable	1
AK	3001-507-800	Scale Keypad Cable	1
AL	23–91	Pan-Hd. Hi-Lo Tap. Screw	4
AM	23–87	Pan-Hd. Hi-Lo Tap. Screw	4
AN	3001-507-910	Scale Keypad Assembly	1
AP	3001-400-953	Switch Cap	6
AQ	3001–507–1	Scale Module	1
AR	3001–507–2	Scale Module Label	1
AS	3001–507–11	Scale Label	1

3001–501–11 Gatch/Fowler Option

Qty. Item Part No. Part Name Part Name Qty. Item Part No. AM 23-87 Pan Hd. Hi-Lo Tap. Screw 3 AA 3001-500-3 Blank Module 1 AT 3000-501-1 Gatch/Fowler Module 1 AB 3000-500-26 Blank Module Label 1 Gatch/Fowler Module Label 1 AU 3001-501-2

Part Name Qty. 3000-500-8 **Chart Rack Cover**

1

3000-525-10 No Chart/Constavac Hanger


3001-507-12 Scale Option Only

ltem	Part No.	Part Name	Qty.
AA	3001–500–3	Blank Module	3
AB	3000-500-26	Blank Module Label	3
AJ	3001-507-900	Scale LCD Display Cable	1
AK	3001–507–800	Scale Keypad Cable	1
AL	23–91	Pan Hd. Hi–Lo Tap. Screw	4
AM	23–87	Pan Hd. Hi–Lo Tap. Screw	4
AN	3001–507–910	Scale Keypad Assembly	1
AP	3001-400-953	Switch Cap	6
AQ	3001–507–1	Scale Module	1
AR	3001–507–2	Scale Module Label	1
AS	3001–507–11	Scale Label	1

3001-507-120 No Scale or Bed Exit Option

ltem	Part No.	Part Name	Qty.
AA	3001-500-3	Blank Module	4
AB	3000-500-26	Blank Module Label	4
AC	3000-500-4	End Module	1
AD	3000–500–27	End Module Label	1

3001-501-110 No Gatch/Fowler Option

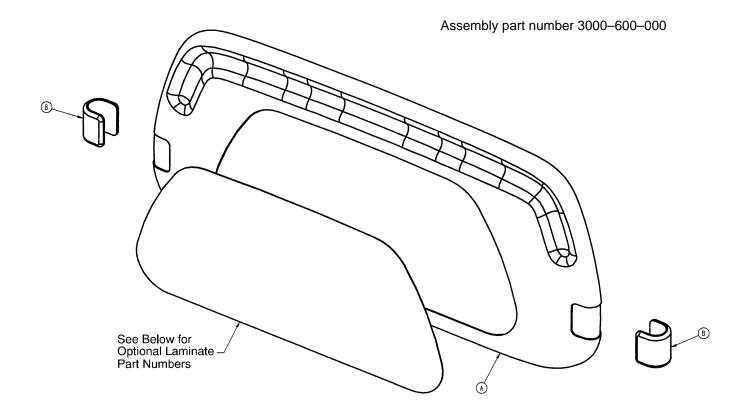


ltem	Part No.	Part Name	Qty.
А	3000-600-10	Clamshell Assembly	1
В	3000-500-7	"C" Bumper	2


3000-600-48

Laminate Color	Part Number
Victorian Cherry	3000–600–50
English Oak	3000–600–49
Pech Contempe Blush	3000–600–45

Oak Suede Classic

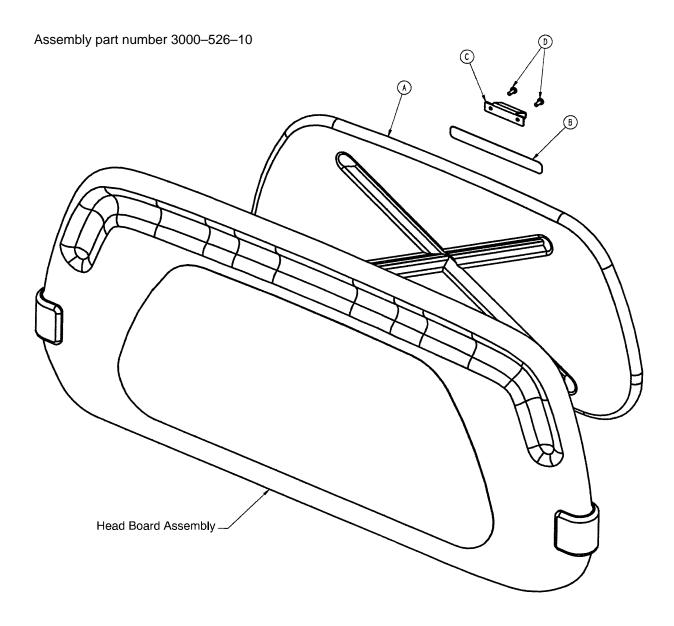


Item	Part No.	Part Name	Qty.
А	3000-526-1	CPR Board	1
В	3000-526-3	CPR Board Label	1
С	3000-526-2	CPR Head Board Clip	1
D	23–88	Pan Hd. Tapping Screw	2

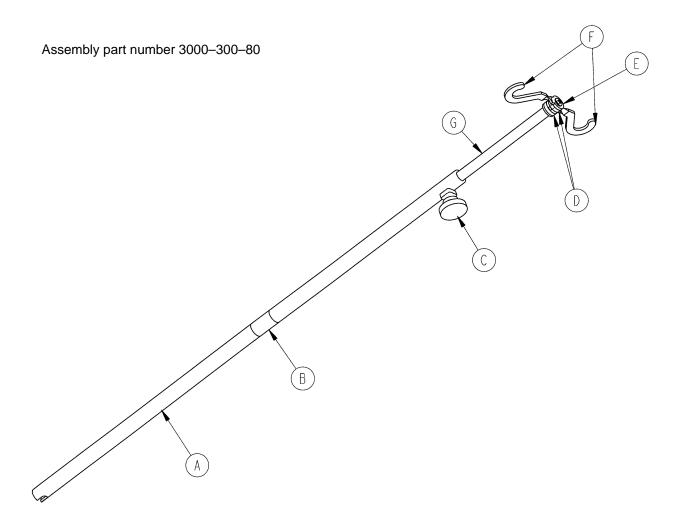


Item	Part No.	Part Name	Qty.
А	3000-300-81	Outer Tube	1
В	3000–300–89	Label	1
С	24–50	Fluted Knob	1
D	52–17	Spacer	2
E	7–40	Phillips Truss Hd. Screw	1
F	1010–59–16	I.V. Hook	2
G	3000–300–85	Inner Tube Assembly	1

Assembly part number 3000–338–10

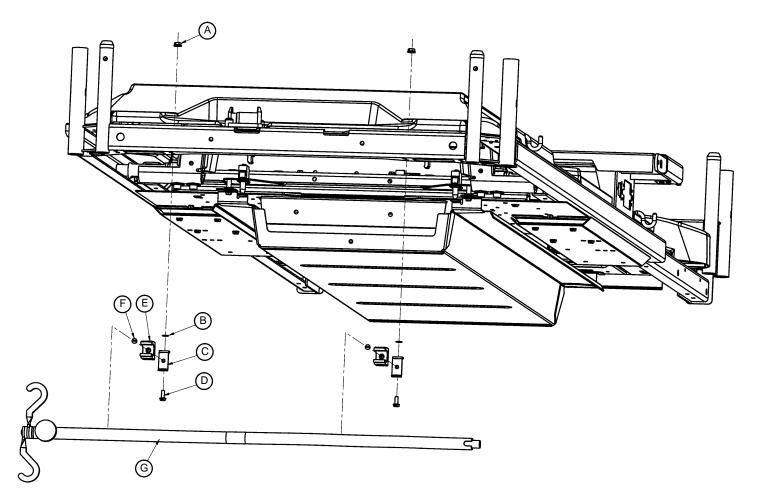


ltem	Part No.	Part Name	Qty.
А	6–74	Flange Nut	2
В	13–18	External Tooth Lock Washer	2
С	3000-300-87	Mounting Bracket	2
D	3–124	Hex Washer Hd. Screw	2
E	958–1–224	I.V. Clip	2
F	2–31	Round Hd. Screw	2
G	(page 198)	I.V. Pole Assembly	1

Permanent I.V. Pole Mounting Assembly, Head End

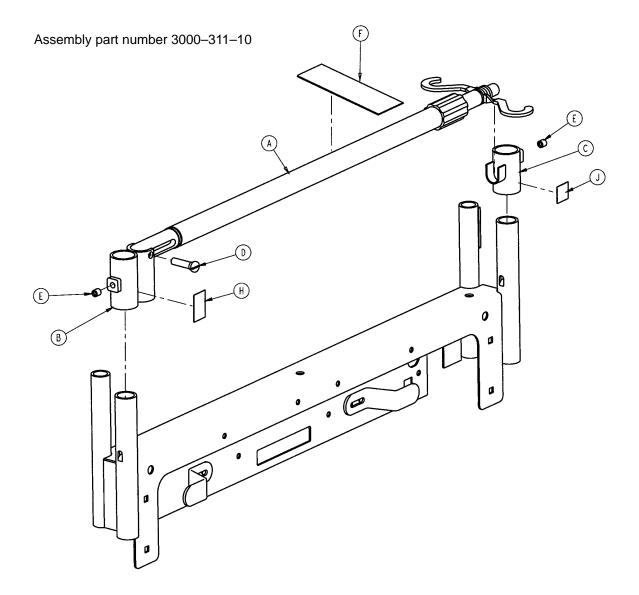

Item	Part No.	Part Name	Qty.
А	(page 202)	I.V. Pole Assembly	1
В	3000–311–11	I.V. Receptacle Assembly	1
С	3000-311-16	I.V. Rest Assembly	1
D	1015–24–35	Retaining Pin	1
E	21–140	Set Screw	2
F	3000–311–5	Spec. Label	1
Н	3000-311-15	Receptacle Label	1
J	3000–311–6	Cradle Label	1


Permanent I.V. Pole Mounting Assembly, Foot End

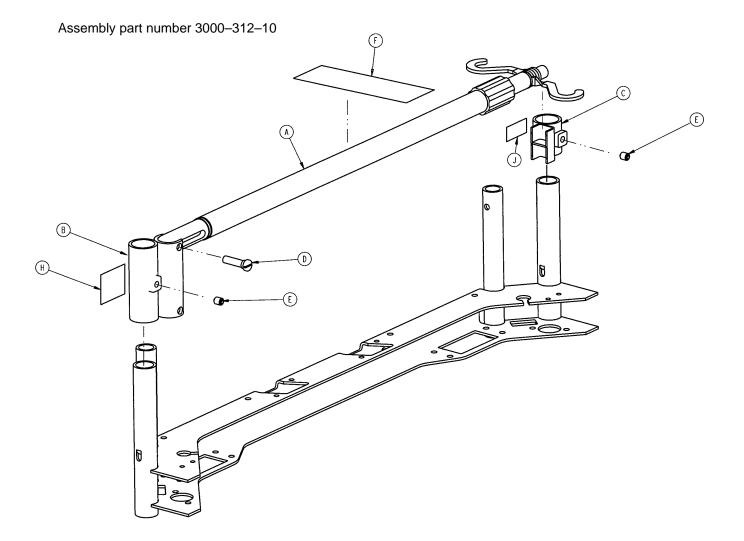

ltem	Part No.	Part Name	Qty.
А	(page 202)	I.V. Pole Assembly	1
В	3000-312-11	I.V. Receptacle Assembly	1
С	3000-312-35	I.V. Cradle Assembly	1
D	1015–24–35	Retaining Pin	1
E	21–140	Set Screw	2
F	3000–311–5	Spec. Label	1
Н	3000-312-7	Receptacle Label	1
J	3000–312–6	I.V. Cradle Label	1

Optional Permanently Mounted I.V. Pole Assembly

Assembly part number 3000–311–26

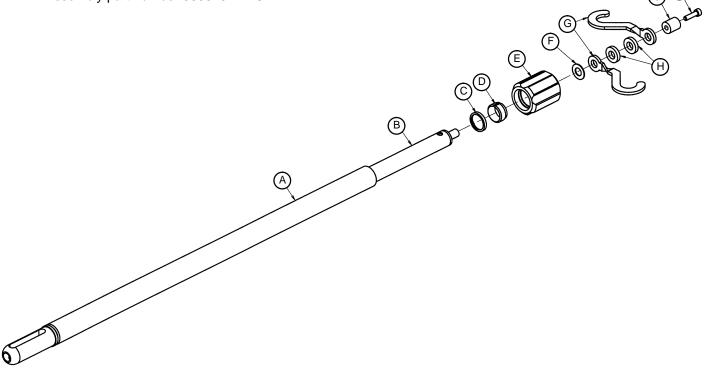


Item A B C D E F G


H J K

Part No.
1010–59–24
1010–59–30
1210–110–46
1210–110–47
1210–110–49
14–20
1010–59–16
52–17
3000–311–19
4–8

Part Name	Qty.
Base Tube Assembly	1
Extension Tube Assembly	1
Back–Up Ring	1
Lock Ring	1
Actuator	1
Nylon Flat Washer	1
I.V. Hook	2
Nylon Spacer	2
End Spacer	1
Socket Hd. Cap Screw	1

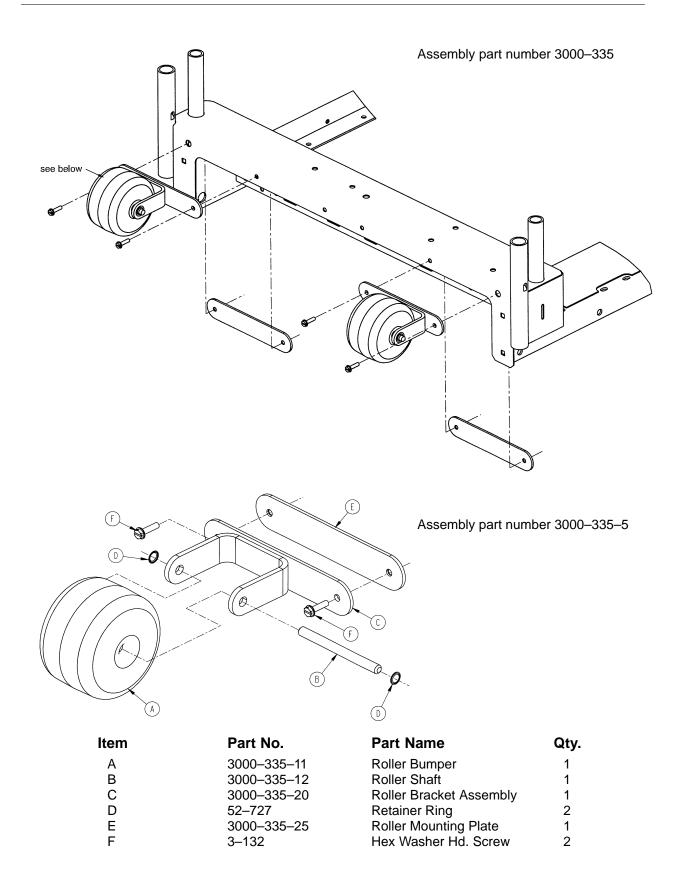


Optional Bed Extender Assembly (Secure Bed Only)

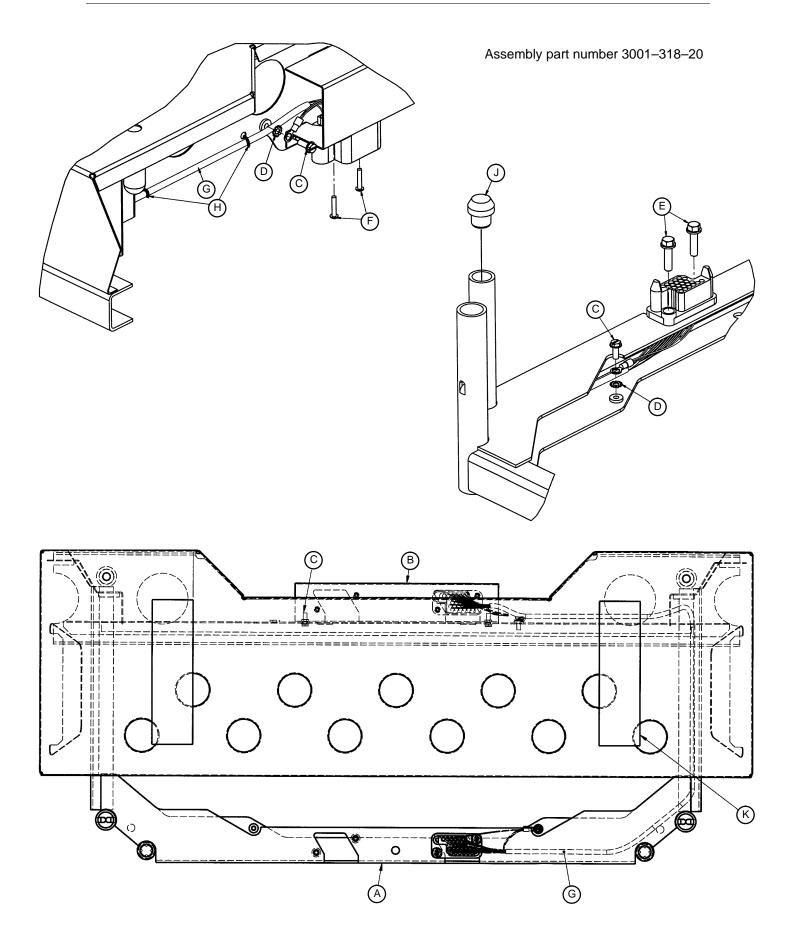

Optional Bed Extender Assembly (Secure Bed Only)


Item	Part No.	Part Name	Qty.
А	3001-318-30	Bed Extender Weldment	1
В	3000-318-21	Connector Mounting Bracket	1
С	3–224	Hex Washer Hd. Screw	4
D	13–18	External Tooth Lock Washer	2
E	3001–200–228	Mounting Standoff	2
F	50–39	Pan Hd. Machine Screw	2
G	3000-318-801	Bed Extender Cable	1
Н	3000-300-114	Cable Tie	2
J	3000-300-349	Cap	2
К	7900–1–102	Velcro Adhesive Pile	2

Optional Bed Extender Assembly (MPS & Secure Bed)

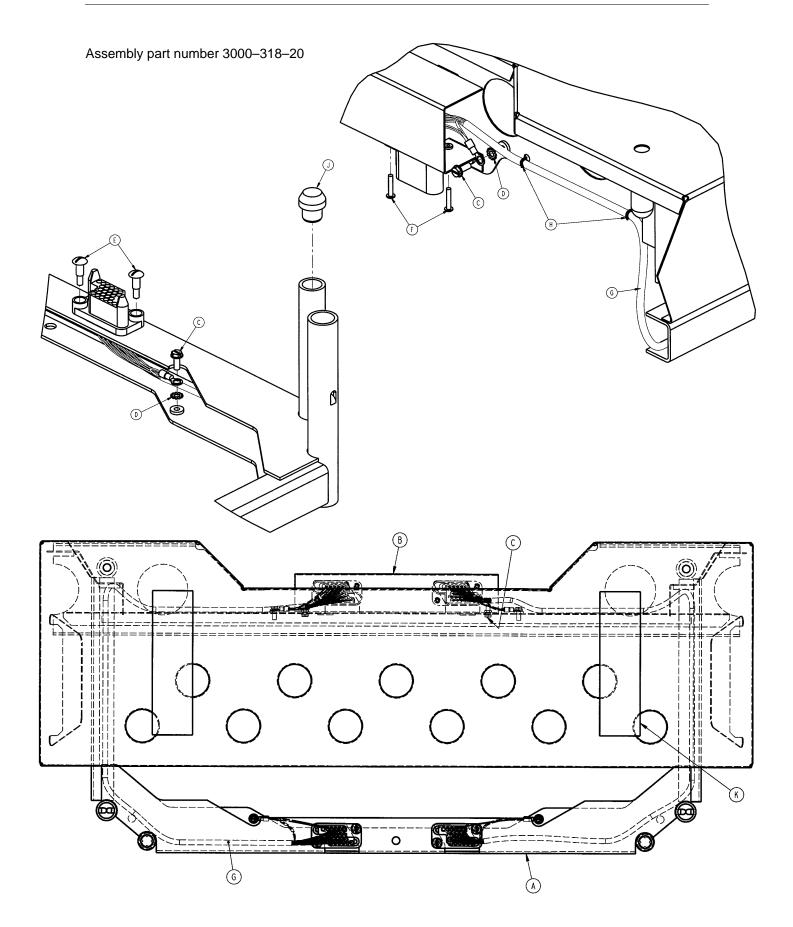


Optional Bed Extender Assembly (MPS & Secure Bed)

Item	Part No.	Part Name	Qty.
А	3000-318-30	Bed Extender Weldment	1
В	3000-318-21	Connector Mounting Bracket	1
С	3–124	Hex Washer Hd. Screw	6
D	13–18	External Tooth Lock Washer	4
E	52–729	Truss Hd. Shoulder Screw	4
F	50–39	Pan Hd. Machine Screw	4
G	3000-318-801	Bed Extender Cable	4
Н	3000-300-114	Cable Tie	4
J	3000-300-349	Cap	2
К	7900-1-102	Velcro Adhesive Pile	2

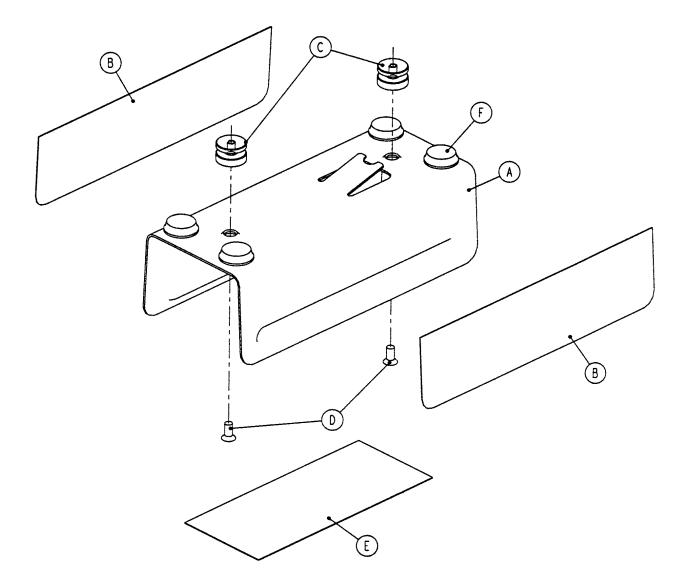

Item	Part No.	Part Name	Qty.
А	3001–305–1	Telephone Bracket	1
В	3000-305-11	Installation Label	2
С	3000-305-13	Mounting Post	2
D	1–22	Flat Soc. Hd. Cap Screw	2
E	3001-305-12	Identification Label	1
F	56–14	Bumper	4


Bed Assembly

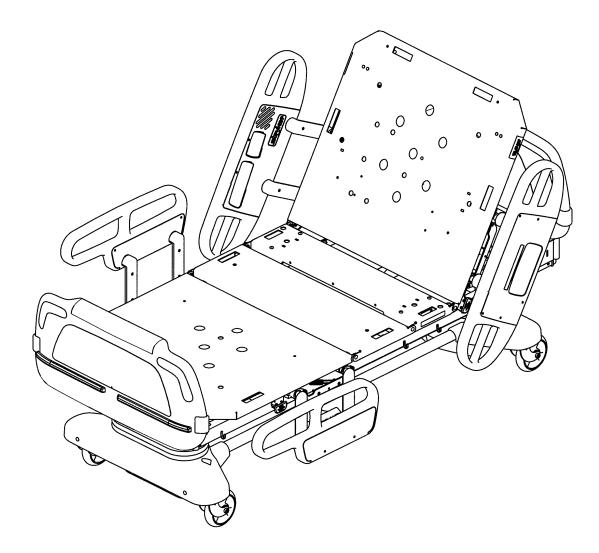

Bed Assembly – Standard Components

Item	Part No.	Part Name	Qty.
А	(page 103–111)	Litter Assembly	1
В	(page 155–178)	Head, Left Siderail Ass'y	1
С	(page 155–178)	Head, Right Siderail Ass'y	1
D	(page 180–182)	Foot, Left Siderail Ass'y	1
E	(page 180–182)	Foot, Right Siderail Ass'y	1
F	3000-600-000	Head Board Assembly	1
G	(page 191–195)	Foot Board Assembly	1
Н	7–52	Truss Hd. Screw	8
J	3000-300-625	Date-of-Manufacture Label	1
K	3–124	Hex Washer Hd. Screw	4
L	16–41	Kep–Nut	4
Μ	3000–300–113	Cable Tie	5
N	3000-300-477	CPR Conduit Stud	3
Р	59–743	Wire Harness Clip	3
Q	3000–300–641	Foot Board Ground Label	1

Bed Assembly

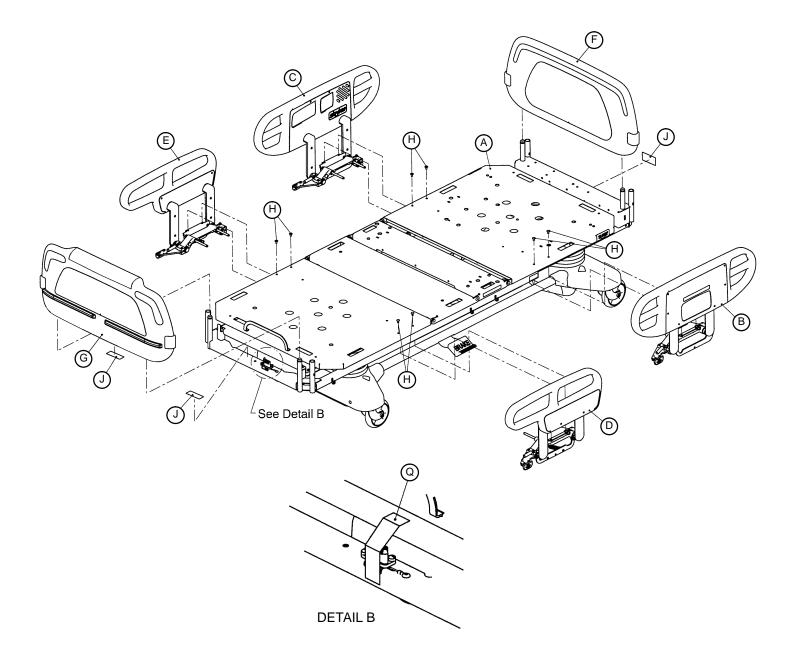

Bed Assembly – 8" Shorter Bed

Bed Assembly - Std. Bed

ltem	Part No.	Part Name	Qty.	ltem	Part No.	Part Name	Qty.
Α	(page 103–111)	Litter Assembly	1	А	(page 103–111)	Litter Assembly	1
В	(page 155–178)	Head, Left Siderail Ass'y	1	В	(page 155–178)	Head, Left Siderail Ass'y	/ 1
С	(page 155–178)	Head, Right Siderail Ass'	y 1	С	(page 155–178)	Head, Right Siderail Ass	s'y 1
D		Foot, Left Siderail Ass'y	1	D		Foot, Left Siderail Ass'y	
Е	(page 180–182)	Foot, Right Siderail Ass'y	1	Е	(page 180–182)	Foot, Right Siderail Ass'	y 1

