



## MAINTENANCE MANUAL

### ELECTRIC MED/SURG BED Model MA204



#### TECHNICAL ASSISTANCE AND PARTS

Canada: 1 888 233-6888

Outside Canada and the United States: Contact your local representative

Manufactured by Stryker

72-0715E R1.0  
F15-44-E-A

May 2004  
Printed in Canada



# TABLE OF CONTENTS

1. INTRODUCTION .....	5
1.1 Specifications .....	5
1.2 Technical Support .....	6
1.3 Warning, Caution, Note Definition .....	6
1.4 Static Discharge Precautions .....	6
1.5 Warranty .....	7
Limited Warranty .....	7
To Obtain Service and/or Parts .....	7
Return Authorization .....	8
Damaged Merchandise .....	8
1.6 Symbols .....	8
2. PREVENTATIVE MAINTENANCE .....	9
2.1 Bed Cleaning and Mattress Care .....	9
Cleaning Beds .....	9
Mattress Care .....	9
2.2 Lubrication Requirements .....	10
2.3 Preventative Maintenance Program .....	12
Annual Checklist .....	12
Recommended Spare Parts .....	13
3. TROUBLESHOOTING .....	15
3.1 Troubleshooting Guide .....	15
4. MAINTENANCE PROCEDURES .....	18
4.1 Siderail Maintenance Procedures .....	18
Head End Siderail Assembly Replacement .....	18
Foot End Siderail Assembly Replacement .....	19
Foot End Rail Replacement .....	20
Head End Rail Replacement .....	20
Siderail Mechanism Transfer Plate Replacement .....	21
Foot End Siderail Mechanism Replacement .....	21
Head End Siderail Mechanism Replacement .....	21
Siderail Control Membrane Replacement .....	22
Siderail Outer Panel Control Board Replacement .....	22
Siderail Inner Panel Control Board Replacement .....	23
Siderail Inner Panel Speakerphone Replacement .....	24
4.2 Foot Board Maintenance Procedures .....	25
Foot End Control Membrane Replacement .....	25
Weigh System LCD Display (Optional) Replacement .....	26
Foot End Control Board Replacement .....	26
4.3 Mattress Support Maintenance Procedures .....	27
Foot Section Replacement .....	27
Thigh Section Replacement .....	28
Seat Section Replacement .....	28

Head Section Replacement .....	29
Fowler Actuator Replacement.....	30
Knee Gatch Actuator Replacement.....	32
Hi-Lo Actuator Replacement.....	33
Motor Control Board Replacement.....	35
Nurse Call/Communications Package Control Board (Optional) Replacement.....	36
Power Connector Replacement .....	37
Power Connector Fuse Replacement .....	37
Scale Control Board/Bed Exit Buzzer (Optional) Replacement .....	38
Toroidal Transformer Replacement (International Series Beds).....	39
Load Cell (Optional) Replacement .....	40
Scale (Optional) Calibration .....	41
Verifying the Scale Accuracy .....	41
Scale Calibration .....	42
Cardiac Chair and Auto Contour Micro Switch Replacement .....	43
Cardiac Chair Micro Switch .....	43
Micro Switch Common to the Cardiac Chair and Auto Contour Mechanism.....	44
Auto Contour Micro Switch .....	44
CPR Micro Switch Replacement.....	45
Micro Switch Attached to the Head Section Lever .....	45
Micro Switch Attached to the Frame. ....	46
CPR Activation Cable Replacement.....	46
CPR Pneumatic Cylinder .....	46
4.4 Base Maintenance Procedures .....	47
Brake/Steer Pedal Replacement.....	47
5th Wheel Caster Replacement .....	48
5th Wheel Assembly Replacement .....	48
5th Wheel Swing Arm Assembly Replacement .....	49
Bed Caster Replacement.....	49
Brake Rod Replacement.....	50
Appendix A: Circuit Diagrams .....	51

## 1. INTRODUCTION

This manual is designed to assist in the servicing of the Stryker's MA204 Med/Surg Beds. Read it thoroughly before beginning any service on the bed. Qualified maintenance personnel should be able to refer to this manual at all time when servicing the bed.

This Maintenance Manual is an integral part of the bed and should be included if the bed is sold or transferred.

### 1.1 SPECIFICATIONS \*

Safe Working Load	500 lb (227 kg)
Weigh System (Optional)	
- Capacity	Patients weighing up to 500 lb (227 kg)
- Accuracy	± 2 % of patient's weight from 100 lb (45.3 kg) to 500 lb (227 kg)
- Operating Angular Range	± 2 lb for patient's weight under 100 lb (45.3 kg) -12° to +12°
Overall Length/Width	
- Siderails Up	93.75 x 42.9" (238 cm x 109 cm)
- Siderails Down	93.5 x 39.3" (237.5 cm x 99.8 cm)
Weight w/Boards	488 lb (221 kg)
Patient Sleep Surface	35 x 80" (89 x 203 cm) extendable to 82" (208 cm) and 84" (213 cm)
Recommended Mattress Size	35 x 80" (89 x 203 cm); 35 x 82" (89 x 208 cm); 35 x 84" (89 x 213 cm)
Mattress <b>Maximum</b> thickness	6" (15.24 cm)
Min/Max Bed Height:	14 to 29" (36 to 73.7 cm)
Fowler Angle	0 to 62°
Knee Gatch Angle	
- W/Auto Contour	0 to 24°
- W/o Auto Contour	0 to 32°
Trendelenburg/Reverse Trendelenburg	+14 to -14°
Environmental Conditions	
- Transport and storage	
- Ambient Temperature	-40 to 70°C (-40 to 158°F)
- Relative humidity	10 to 100%
- Atmospheric Pressure	500 to 1060 hPa
- Operating **	
- Ambient Temperature	18.3 to 26.7°C (65 to 80°F)
- Relative humidity	20 to 80% without <b>condensation</b>
- Atmospheric Pressure	700 to 1060 hPa
*** Electrical Requirements - all electrical requirements meet CSA C22.2 No. 601.1-M90, UL 60601 1 and IEC 60601-1, 60601-2-38 specifications.	100V~, 50-60Hz, 7.5A - Two 250V, 10A Fast Acting Fuses 120V~, 50-60Hz, 4.0A (9.8A w/120V Optional Auxiliary Outlet) - Two 250V, 10A Fast Acting Fuses 200V~, 50-60Hz, 3.2A -Two 250V, 6.3A Slow Blow Fuses 220V~, 50-60Hz, 2.9A -Two 250V, 6.3A Slow Blow Fuses 240V~, 50-60Hz, 2.7A -Two 250V, 6.3A Slow Blow Fuses

\* Stryker pays special attention to product improvement and reserves the right to change specifications without notice.

\*\* Operating environment recommended to ensure the scale precision.

\*\*\* The device has a 10% duty cycle.

## 1.2 TECHNICAL SUPPORT

For questions regarding this product, contact the following Technical Service department or your local representative:

**Stryker Canada**

1 888 233-6888

45, Innovation Drive

Hamilton, Ontario, L9H 7L8

## 1.3 WARNING, CAUTION, NOTE DEFINITION

The words WARNING, CAUTION and NOTE carry special meanings and should be carefully reviewed.



### WARNING

The personal safety of the patient or user may be involved. Disregarding this information could result in injury to the patient or user.



### CAUTION

These instructions point out special procedures or precautions that must be followed to avoid damaging the equipment.

### NOTE

Notes provide special information to make maintenance easier or important instruction clearer.

## 1.4 STATIC DISCHARGE PRECAUTIONS


The electronic circuits of the bed are protected from static electricity damage only while the bed is assembled. It is extremely important that all service personnel always use adequate static protection when servicing the electronic components of the bed.

### Static Protection Equipment

The necessary equipment for a proper static protection is:

- 1 static wrist strap
- 1 grounding plug
- 1 test lead with a banana plug on one end and an alligator clip on the other.

### Static Protection Procedure

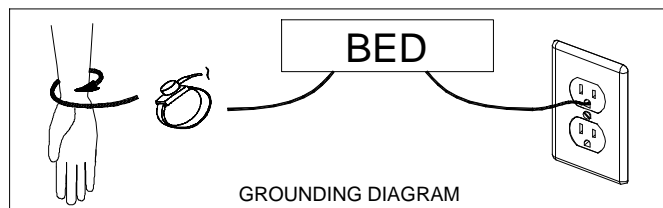

1. Unplug the bed power cord from the wall outlet.
2. Insert the grounding plug into a properly grounded hospital grade wall receptacle. Plug the banana plug of the test lead into the receptacle on the grounding plug. Connect the alligator clip on the other end of the test lead to a ground point on the bed.

3. Place the static control wrist strap on your wrist. Connect the clip at the other end of the wrist strap cord to a ground point on the bed.

## 1.5 WARRANTY

### LIMITED WARRANTY

All Stryker products are guaranteed against material or manufacturing defects, improper operation of mechanisms, and premature wear of bed components under normal use conditions.

For questions regarding warranty, please contact the Technical Service department (see section 1.2) or your local representative.


### TO OBTAIN SERVICE AND/OR PARTS

- **To Require Service**

For an on-site diagnosis of a malfunction by one of our Filed Service Representative, contact the Technical Service department (see section 1.2) or your local representative.

- **To Order Parts**

In order to correctly identify and order parts to be replaced, proceed as follows:

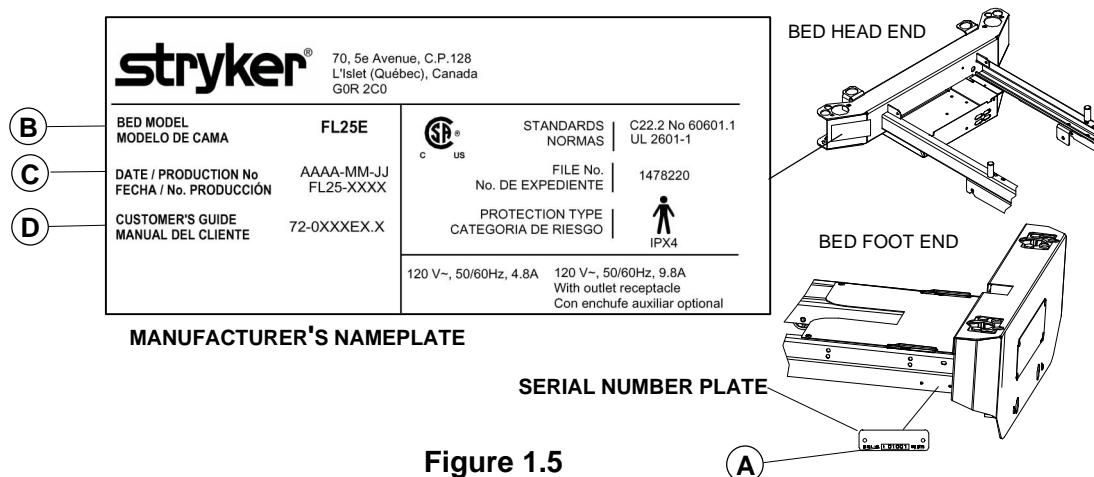

- Locate the serial number plate and the manufacturer's nameplate affixed respectively to the right side of the frame at the foot end of the bed, and on the right side of the head end casing.
- From the serial plate, write down the serial number (A).
- From the manufacturer's nameplate, write down the bed model (B), the production number (C), ex. FL25-XXXX, and the Customer's Guide number (D).
- Consult the parts lists and the drawings contained in the Parts List Manual included in the Customer's Guide to identify the defective part. Write down the name of the part and its part number, and the problem encountered while using the equipment.

#### NOTE

It is very important that you refer to the parts lists and drawings of the Parts List manual specific to the bed needing to be repaired.

We will do our best to help you identify the parts to be replaced. However, if an error occurs when ordering, the user remains responsible for identifying the parts to change. Stryker will take back wrong parts ordered but will not assume shipping charges, and restocking fees will be charged to the user unless a Technical Service Representative has been requested for an on-site diagnosis of the malfunction.

**RETURN AUTHORIZATION**

Merchandise cannot be returned without approval from the Technical Service department. An authorization number will be provided, which must be clearly printed on the returned merchandise. Stryker reserves the right to charge shipping and restocking fees on returned items.

**DAMAGED MERCHANDISE**

Claims for damaged merchandise must be made with the carrier within fifteen (15) days of receipt of merchandise. **DO NOT ACCEPT DAMAGED SHIPMENTS UNLESS SUCH DAMAGE IS NOTED ON THE DELIVERY RECEIPT AT THE TIME OF RECEIPT.** Upon prompt notification, Stryker will file a freight claim with the appropriate carrier for damages incurred. Claims will be limited in amount to the actual replacement cost. In the event that this information is not received by Stryker within the fifteen (15) day period following the delivery of the merchandise, or the damage was not noted on the delivery notice at the time of receipt, the customer will be responsible for payment of the original invoice in full.

Claims for any short shipment must be made within 5 days of invoice.

**1.6 SYMBOLS**

Warning, refer to Maintenance Manual



Fuse rating for bed with the 100V~ and 120V~ electric systems



Fuse rating for beds with 200V~, 220V~ and 240V~ electric systems



Protective earth (ground)



Alternating Current



Type B Equipment

**IPX4** Protection from liquid splash



## 2. PREVENTATIVE MAINTENANCE

### 2.1 BED CLEANING AND MATTRESS CARE

**WARNING**

Always unplug the bed power cord from the wall outlet when cleaning or servicing the bed.

**CAUTION**

Do not use harsh cleaners, solvents or detergents. Equipment damage could occur. Do not steam clean, hose off or ultrasonically clean the bed. Do not immerse any part of the bed. The bed electrical parts may be damaged by exposure to water.

Germicidal disinfectant, used as directed, and/or Chlorine Bleach products are not considered mild detergents. These products are corrosive in nature and may cause damage to your bed if used improperly. If these types of products are used, ensure the beds are wiped with clean water and thoroughly dried following cleaning. Failure to properly rinse and dry the beds will leave a corrosive residue on the surface of the bed, possibly causing premature corrosion of critical components. Failure to follow the above directions when using these types of cleaners may void this product warranty.

**CLEANING BEDS**

Hand wash all surfaces of the bed with a soft cloth moistened with a solution of lukewarm water and a mild detergent.

Wipe the bed clean and dry thoroughly to avoid build up of cleaning solution.

**MATTRESS CARE****WARNING**

Inspect the mattress after each use. Discontinue use if any cracks or rips, which may allow fluid to enter the mattress, are found in the mattress cover. Failure to properly clean the mattress, or dispose of it if defective, may increase the risk of exposure to pathogenic substances and may bring about diseases to the patient and user.

- **Inspection**

Implement local policies to address regular care, maintenance, and cleaning of mattresses and covers. The cover cleaning procedure can be found below and on the bed label.

Inspect mattress cover surface (also zip fasteners and cover inner surface if mattresses have zip fasteners) regularly for signs of damage. If the mattress cover is heavily stained or soiled, or is torn, remove the mattress from service.

- **Cleaning**

Stains: Wash with lukewarm water using a mild detergent. Rinse with water and let dry. For tough stains use bleach diluted with ten parts of water.

## 2.2 LUBRICATION REQUIREMENTS

The only components of the bed needing periodic lubrication are the four actuator screws. They should be checked every year and lubricated every two years.



### CAUTION

The bed uses oil-impregnated shoulder spacers at hinge points. **Do not** lubricate these shoulder spacers. When shoulder spacers are found worn, replace them.

### ACTUATOR SCREW LUBRICATION

#### Required Tools:

Phillips Screwdriver

OG2 Grease

1/4" Ratchet (w/6" extension) and 5/16" socket

Brush

#### Procedure:

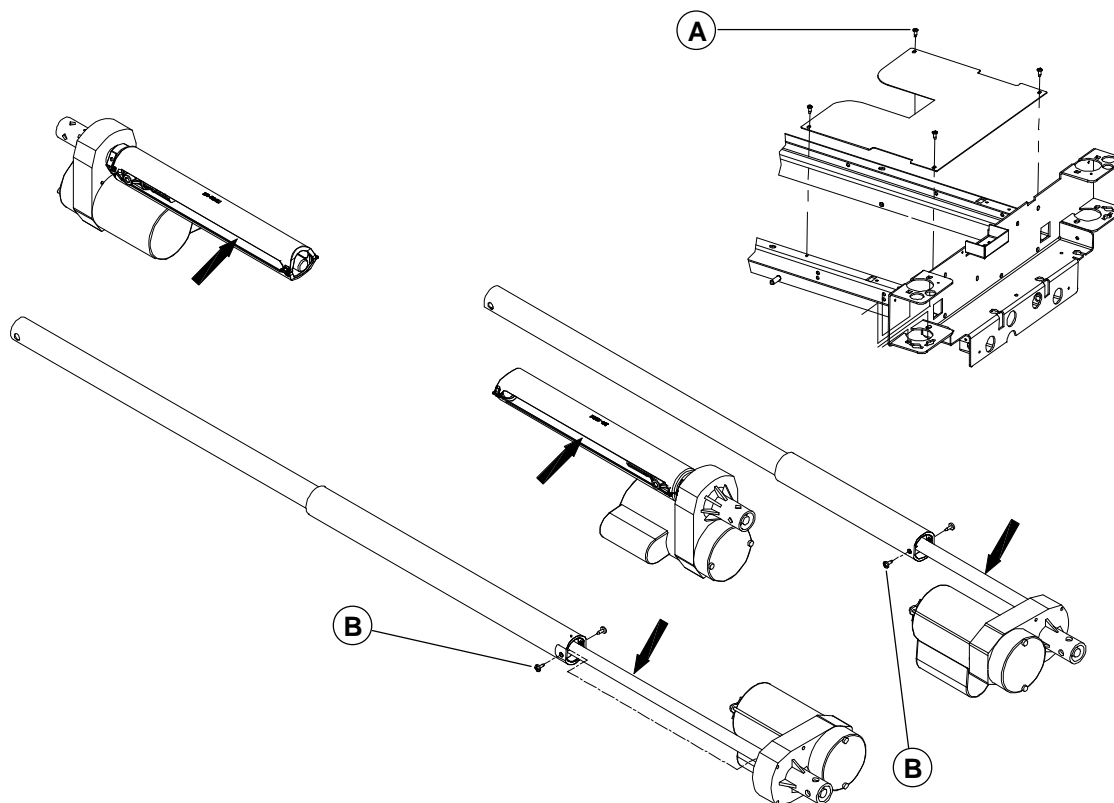


Figure 2.2

#### Head and Thigh Actuators

1. Raise the bed fully up and apply the brakes.
2. Remove the plastic molded shells from the head and foot sections.
3. Fully raise the head section and flatten the thigh section to expose maximum screw threads on both actuators.
4. Unplug the power cord from the wall outlet.
5. Lift and fold back the foot section toward the head end of the bed.
6. Using a Phillips screwdriver, remove the four screws (A) holding the cover plate to the frame and remove the plate. Remove the optional night light if present.
7. Using a 1/4" ratchet (w/6" extension) and a 5/16" socket, remove the two screws (B) holding the dust tube of each actuator.

8. Using a brush, apply grease on the screw threads. Make sure the grease reaches the bottom of the threads.
9. Replace the dust tubes.
10. Using the electric controls, raise and lower several times the Fowler and Knee Gatch to spread grease evenly.
11. Replace the cover plate.

**Hi-Lo Actuators**

12. Lower the bed completely and apply the brakes. Flatten the mattress support.
13. Unplug the power cord from the wall outlet.
14. Remove the plastic molded shells from the head and foot sections.
15. Using a brush, apply grease on the threads through the dust tube side openings of both Hi-Lo actuators. Make sure the grease reaches the bottom of the threads.
16. Using the electric controls, raise and lower the bed several times to spread the grease evenly.

## 2.3 PREVENTATIVE MAINTENANCE PROGRAM



### WARNING

When servicing use only identical replacement parts provided by Stryker.

---

### ANNUAL CHECKLIST

- All fasteners secure.
- Inspect for excessive wear the oil-impregnated bronze shoulder spacers found at the bed hinge points. Replace as needed. **Do not** lubricate these spacers.
- Check the grease present on the actuator screws, lubricate if needed (see the screw lubrication procedure on page 10). The actuator screws need lubrication every two years.
- On both sides of the bed, depress fully down the side of the pedal identified with a red sticker and ensure that the brakes are applied and the bed immobilized. Toggle the pedal to neutral (horizontal position) and ensure the brakes are released.
- On both sides of the bed, depress fully down the side of the pedal identified with a green sticker and ensure that the 5th steer wheel is engaged. Toggle the pedal to neutral (horizontal position) and ensure that the 5th wheel disengages.
- Siderails move, latch and stow properly.
- All controls of the foot board panel working properly, including LED's.
- Calibrate the optional scale (see the scale calibration procedure on page 42).
- All siderail controls working properly.
  - Ensure that the nurse call alarm sounds in the nurse station.
  - Ensure that the communications package controls operate properly.
- Fowler and Knee Gatch (if raised) flatten and the Fowler control motor resets itself automatically (as soon as the fowler is completely lowered) when both CPR handles are pulled.
  - Wait about 30seconds, the time for the Fowler motor to reset itself, and then raise the Fowler to ensure that the resetting of the motor has indeed occurred.
- Verify the Fowler, Knee Gatch and Hi-lo movements to ensure that the motion interrupt switch integrated to the four electric actuators is operating properly.
- Auxiliary outlet (option available only with 120V beds) working properly.
- Night light (option) working properly.
- No cracks in the boards, siderails, wheel covers, 5th wheel hood (optional) and molded shells of the head and foot sections. Velcro strips on shells and structures match properly.
- Head end bumpers tightly secured to frame and working properly.
- No rips or cracks in mattress cover.
- Power cord not frayed.
- No cables worn or pinched.
- All electrical connections tight.
- All grounds secure to the frame.
- All casters roll properly. Check caster for cuts, wear, etc.
- Measure current leakage and grounding continuity of the bed and the optional auxiliary outlet. Check with our Technical Service department for the acceptable values.

**RECOMMENDED SPARE PARTS**

The following is a list of recommended on hand spare parts for the MA204 bed.

<b>Description</b>	<b>Part Number</b>	<b>Parts List</b>
<b>• Electric/Electronic Components</b>		
Motor Control Board	QDF25-0304	OL250024
Scale Control Board (Optional)	25-0492	OL250041
Nurse Call/Gen III Control Board (Optional)	QDF21-1178	OL250082
Foot End Control Board	QDF21-1154	OP250001
Foot Board Interface Plug	QDF21-2898	OL250001
Micro Switch	1325P003	L25-017
Push Button Micro Switch	QDF9159	L25-017
Power Connector	QDF2034	OL250024
Fuse (Fast Acting 10A, 250V for 100/120V Electric System)	QDF8078	OL250024
Fuse (Slow Blow 6.3A, 250V for 200/220/240V Electric System)	QDF8068	OL250065/ 071/076
Power cord with Straight N A Molded Plug	QDF8066	OL250053
Power cord with 90° N A Molded Plug (Optional)	QDF8066-90D	OL250055
Network Y Cable	QDF25-0386	OL250024
Speakerphone w/Wires (Optional)	QDF25-0383	OL250107
Night Light (Optional)	QDF9539	OL250018
Load Cell w/Long Cable (Optional)	QDF20-0014	OL250041
Load Cell w/Short Cable (Optional)	QDF25-0218	OL250041
Auxiliary Power Outlet (Optional)	QDF8024	OL250029
5A Circuit Breaker (for Optional Auxiliary Power Outlet)	QDF9025	OL250029
Hi-Lo Actuator	QDF20-0247	OL250024
Head Section Actuator	23-0249	OL250024
Thigh Section Actuator	QDF20-0248	OL250024
Toroidal Transformer (International Series Bed)	14-1160	OL250063/ 065/071/076
Stand Off Pins: Motor Control Board/Scale Control Board	QP23-0258/ QDF8011	OL250024/ OL250041
Strain Relief Bushing	QDF9541	OL250024
<b>• Foot Board Components</b>		
Control Panel Membrane w/Scale and Bed Exit 3 Zone	25-0461	OP250002
Control Panel Membrane w/Scale and Bed Exit 1 Zone	25-0528	OP250003
Control Panel Membrane w/Scale w/o Bed Exit	25-0529	OP250004
Control Panel Membrane w/o Scale and w/o Bed Exit	25-0530	OP250005
Control Panel Membrane w/o Scale and w/Bed Exit 1 Zone	25-0531	OP250006
Scale LCD Display	QDF25-0463	OP250001
Round Screw Cover	QDF21-3943	OP250001
Head/Foot Board Leg cap	QPC21-3855	OP250001
<b>• Siderail Components</b>		
Inner Right Ctrl Panel Membrane: w/o Nurse Call nor Com. Package	QDF25-0534	OL250080
Inner Left Ctrl Panel Membrane: w/o Nurse Call nor Com. Package	QDF25-0535	OL250080

Inner Ctrl Panel: No Speakerphone Label	QDF21-4354	OL250080
Inner Ctrl Panel: Speakerphone Label	21-1868	OL250107
Inner Right Ctrl Panel Membrane: w/Nurse Call, w/o Com. Package	QDF25-0532	OL250107
Inner Left Ctrl Panel Membrane: w/Nurse Call, w/o Com. Package	QDF25-0533	OL250107
Inner Right Ctrl Panel Membrane: w/Nurse Call and Com. Package	QDF25-0389	OL250112
Inner Left Ctrl Panel Membrane: w/Nurse Call and Com. Package	QDF25-0390	OL250112
Outer Right Ctrl Panel Membrane	QDF25-0044	L25-014
Outer Left Ctrl Panel Membrane	QDF25-0045	L25-014
Outer Control Board	QDF21-1151	L25-014
Inner Control Board	QDF21-1169	L25-014
"Lift to Release" Label	QDF21-3680	L25-014
Oblong Screw Cover	QDF25-0025	L25-014
Left Transfer Plate	25-0523Z	L25-014
Right Transfer Plate	25-0524Z	L25-014
Nylon Stopper	20-0197	L25-014
Siderail Mechanism Spring	QRE21-4384	L25-014

- Mattress Support Components**

Head Section Plastic Molded Shell	QP25-0054	L25-003
Foot Section Plastic Molded Shell	QP25-0038	L25-003
Seat Section	23-0239P	L25-003
Thigh Section	25-0163P	L25-003
Adjustable Mattress Retainer	17-0211P	L25-003
Rubber Cap	QPCF1001	L25-008
CPR Pneumatic Cylinder	QDF5090	L25-019
CPR Right Handle Cable	QDF19-0815	L25-019
CPR Left Handle Cable	QDF19-0354	L25-019
Velcro Strip for the Molded Shell	25-0302	L25-003

- Hi-Lo Mechanism Components**

Right "Stub-Acme" Nut	QP13-0677-05	OL250024
Nut Support	QPA25-0380	OL250024

- Base Components**

6" Caster w/Locking Mechanism	R25-0388	OL250011/ OL250012
6" Caster w/o Locking Mechanism	R25-0493	OL250011
Directional 5th Wheel	RL5	OL250045
Wheel Cover Right	QP25-0022	L25-004
Wheel Cover Left	QP25-0039	L25-004
Brake/Steer Pedal	21-3761	OL250045

- Miscellaneous**

OG2 Grease	M0027	
White Spray Paint	DDCAP-BLP	
Nylon Cable Tie	QDF9521	

### 3. TROUBLESHOOTING

Please consult the following troubleshooting guide before calling the Technical Service department (see section 1.2).

#### 3.1 TROUBLESHOOTING GUIDE

PROBLEM/FAILURE	WHAT TO CHECK
No power to bed	<ul style="list-style-type: none"> <li>• Is the bed power switch at the head end of the bed turned on?</li> <li>• Is the power cord connected to the power connector and plugged into the wall outlet?</li> <li>• Is the power cord severed? Replace if needed.</li> <li>• Are the two fuses inside the power connector still operational (see page 37)</li> <li>• Verify power at wall outlet.</li> </ul>
No bed up or down motion when: <ul style="list-style-type: none"> <li>• the siderail command is used</li> <li>• the foot board command is used</li> </ul>	<ul style="list-style-type: none"> <li>• Is the total lockout activated (LED (padlock) on) ? If so, deactivate it.</li> <li>• Is the cable of the siderail control panel properly connected to the bed receptacle under the mattress support?</li> <li>• Check points of the “No power to bed” problem described above.</li> </ul>
No Fowler up or down motion when: <ul style="list-style-type: none"> <li>• the siderail command is used</li> <li>• the foot board command is used</li> </ul>	<ul style="list-style-type: none"> <li>• Is the Fowler lockout activated (LED (padlock) on) ? Deactivate it.</li> <li>• Is the total lockout activated (LED (padlock) on) ? If so, deactivate it.</li> <li>• Is the cable of the siderail control panel properly connected to the bed connector under the mattress support?</li> <li>• Check points of the “No power to bed” problem described above.</li> </ul>
The Fowler does not <b>fully</b> raise	<p>This situation happens when the CPR handle is used to partly lower the Fowler. The use of the CPR mechanism for this purpose creates a situation where the Fowler motor is out of sync with the actual position of the Fowler. To correct the situation:</p> <ul style="list-style-type: none"> <li>• Completely lower the Fowler using the CPR handle or the Fowler down control to enable the Fowler motor to reset itself.</li> </ul> <p>Refer to the "Emergency CPR Release" section of the Operations Manual for more information.</p>

<p>No Knee Gatch up or down motion when:</p> <ul style="list-style-type: none"> <li>the siderail command is used</li> <li>the foot board command is used</li> </ul>	<ul style="list-style-type: none"> <li>Is the Knee Gatch lockout activated (LED (padlock) on) ? If so, deactivate it.</li> <li>Is the total lockout activated (LED (padlock) on) ? If so, deactivate it.</li> <li>Is the cable of the siderail control panel properly connected to the bed connector under the mattress support?</li> <li>Check points of the "No power to bed" problem described above.</li> </ul>
No Auto Contour motion	<ul style="list-style-type: none"> <li>Is the Knee Gatch or total lockout activated (LED (padlock) on) ? If so, deactivate it.</li> <li>Check and replace if needed (see page 44) the two Auto Contour limit switches.</li> </ul>
Improper operation of the cardiac chair: Fowler and Knee Gatch raise but final step of the cardiac chair position not reached (reverse Trend).	<ul style="list-style-type: none"> <li>Check and replace if needed (see pages 44 and 45) the two Cardiac Chair limit switches.</li> </ul>
Improper operation of the CPR positioning: Knee Gatch does not lower and/or the Fowler actuator does not reset.	<ul style="list-style-type: none"> <li>Check and replace if needed (see pages 46 and 47) the two CPR limit switches.</li> </ul>
The weight shown on the scale display is incorrect.	<ul style="list-style-type: none"> <li>Zero the bed without the patient in it (see the "Weigh System Usage" section in the Operations Manual).</li> <li>Calibrate the scale (see page 42).</li> </ul>
Scale display reads OVERLOAD or shows incorrect weight.	<ul style="list-style-type: none"> <li>The weight present on the bed is superior to the scale capacity, that is 500 lb (227 kg). If the patient weight is superior to 500 lb (227 kg), stop using the scale. If not, zero the scale after having removed the patient from the bed. This situation often results from equipment added to the bed without using the add/remove equipment procedure. The weight of this equipment is added then to the patient's weight.</li> <li>One or more load cell cable not properly connected or completely disconnected from the scale control board (see figure 4.3E, page 38 for the connecting positions of the load cell cable).</li> </ul>
Angle shown on the scale display is incorrect	<ul style="list-style-type: none"> <li>Calibrate the scale (see page 42).</li> </ul>



Scale display does not turn on when the scale is activated.	<ul style="list-style-type: none"> <li>• Press ENTER key twice.</li> <li>• Turn the main power switch off and turn it on.</li> <li>• Lift the foot board and gently replace it into position taking care to completely push it down so that the connectors match correctly.</li> <li>• Check points of the “No power to bed” problem described above.</li> <li>• Scale user interface cable not properly connected or completely disconnected from the scale control board (see figure 4.3E, page 38 for the illustration of the cable connecting position).</li> </ul>
Scale displays "No scale found".	<ul style="list-style-type: none"> <li>• Press the ENTER key. If message persists, contact our Technical Service (see section 1.2). If message disappears, data displayed is correct and current procedure can be resumed.</li> </ul>
All lockout LEDs (padlock icons) flash <u>or</u> the word Error appears at the bottom of the LCD display.	<ul style="list-style-type: none"> <li>• The scale is unable to measure the weight due to an electrical problem. Please contact our Technical Service (see section 1.2).</li> </ul>
Bed Exit system does not operate properly.	<ul style="list-style-type: none"> <li>• Verify that the load cell cables are properly connected to the scale control board (see figure 4.3E, page 38) for the illustration of the load cell cable connecting positions).</li> </ul>
Bed Exit system does not work at all or alarm goes off when Bed Exit is activated.	<ul style="list-style-type: none"> <li>• Check points of the “No power to bed” problem described above.</li> <li>• One or more load cell cable are not properly connected or are completely disconnected from the scale control board (see figure 4.3E, page 38 for the load cell cable connecting positions).</li> </ul>
Nurse call or Bed Exit signal does not reach the nurse desk.	<ul style="list-style-type: none"> <li>• Check points of the “No power to bed” problem described above.</li> <li>• Check the wall outlet.</li> </ul>

## 4. MAINTENANCE PROCEDURES



### WARNING

Only field technicians from Stryker or service personnel trained by Stryker should perform the procedures detailed in this maintenance guide, especially those related to the Scale (option) and Bed Exit systems (option). Failure to observe this restriction can result in serious damage to material and/or severe injury to people.

To prevent injury when working under the bed with the bed in the high position, always place blocks under the mattress support frame and apply the brakes.

Always unplug the bed power cord from the wall outlet when servicing or cleaning the bed.

### NOTE

Throughout this operations manual, the words “right” and “left” refer to the right and left sides of a patient lying face up on the bed.

## 4.1 SIDERAIL MAINTENANCE PROCEDURES


### HEAD END SIDERAIL ASSEMBLY REPLACEMENT

#### Required Tools:

1/2" Socket Wrench (w/6" Extension)

Torque Wrench

#### Procedure:

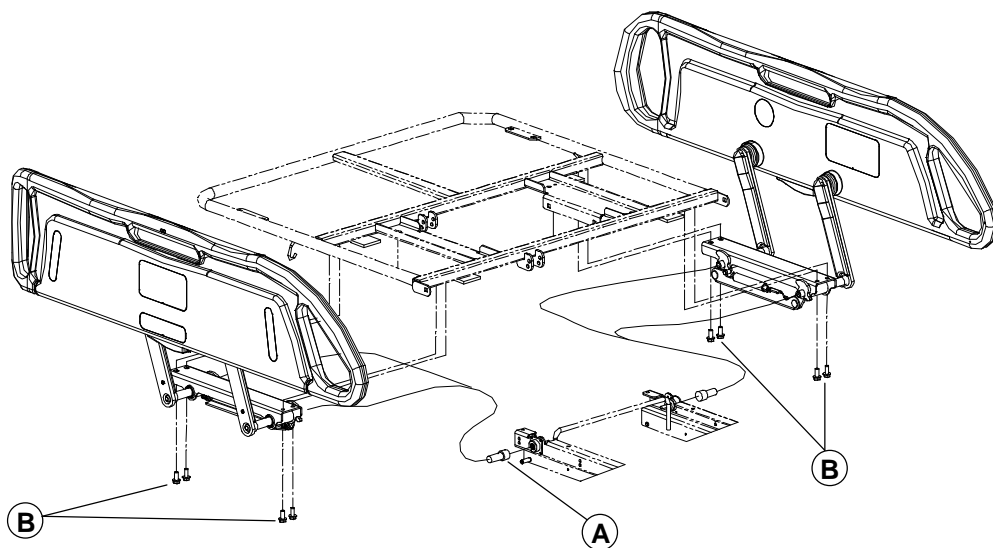

**Figure 4.1A**

1. Run the bed fully up and apply the brakes.
2. Unplug the power cord from the wall receptacle.
3. Remove the molded shell of the head section shell and raise the siderail needing repair.
4. Loosen the lock ring (A) and unplug the siderail control cable.
5. Using a 1/2" socket wrench, remove the four thread-rolling bolts (B) holding the siderail assembly to the head section and remove the assembly. Support the assembly when removing the last bolts.

### NOTE


Once removed, the thread-rolling bolts must be first be screwed in manually before tightening them to a 130 lbf in (14.7 N m) torque using a torque wrench.

6. Reverse the above steps to install the new siderail assembly.
7. Test the siderail movement and all controls of both control panels, including the optional nurse call function, for proper operation before returning the bed to service.

**FOOT END SIDERAIL ASSEMBLY REPLACEMENT****Required Tools:**

1/2" Socket Wrench (w/6" Extension)

Torque Wrench

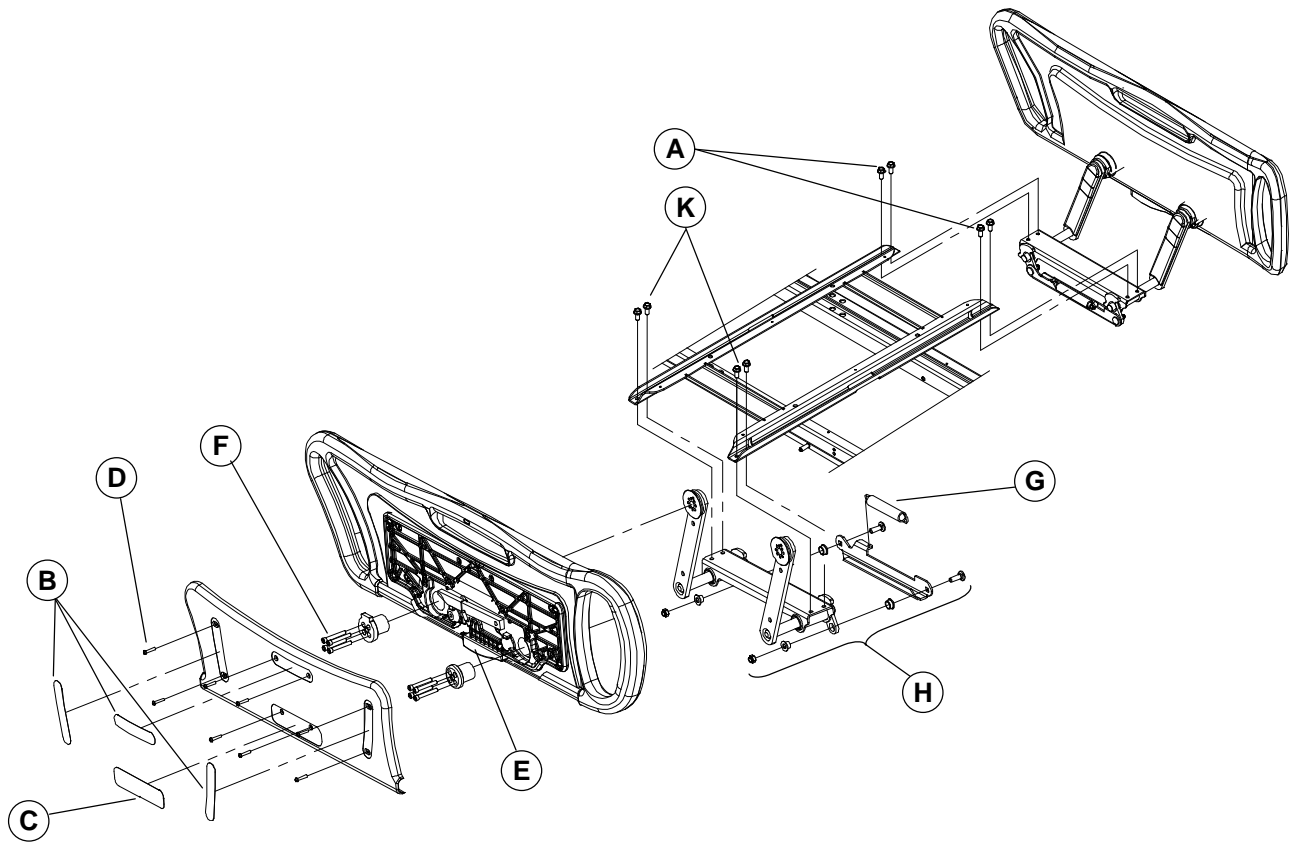**Procedure:****Figure 4.1B**

1. Run the bed fully up and apply the brakes. Raise the siderail needing repair.
2. Run the Knee Gatch fully up and unplug the power cord from the wall receptacle.
3. Lift and fold the foot section back toward the head end of the bed.
4. Using a 1/2" socket wrench, remove the four thread-rolling bolts (A) holding the siderail assembly to the support and remove the assembly. Support the siderail assembly while removing the last bolts.

**NOTE**

Once removed, the thread-rolling bolts must be first be screwed in manually before tightening them to a 130 lbf in (14.7 N m) torque using a torque wrench.

5. Reverse the above steps to install the new siderail assembly.
6. Test the siderail for proper operation before returning the bed to service.

## FOOT END RAIL REPLACEMENT

### Required Tools:


Small Slotted Head Screwdriver  
Bungee Cord

Phillips Screwdriver                      3/16" Allen Key  
Medium-Strength Thread Locker

### Procedure:

#### NOTE

Unless otherwise indicated, refer to figure 4.1B on page 19 for the illustration of this procedure reference points.

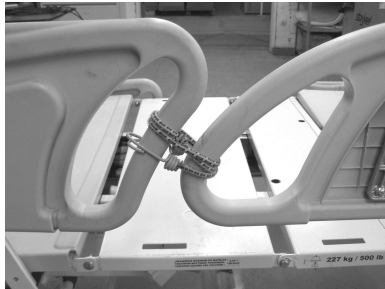

1. Run the bed fully up and apply the brakes. Raise the siderail needing repair and secure it to the adjacent head siderail (raise it too) using a bungee cord (see opposite illustration).
2. Unplug the power cord from the wall receptacle.
3. Using a small slotted head screwdriver, lift and remove the following self-sticking parts: the three screw-covers (D) and the "Lift to release siderail" label (E) to expose the screws holding the siderail cover. Proceed gently when inserting the screwdriver under the labels to avoid scratching the molded rail.

#### NOTE

Do not reuse the self-sticking parts removed since their self-adhesive coating considerably loses its efficiency once they are removed. We recommend that you have replacement stickers at hand (see Recommended Spare Parts on page 13).

4. Using a Phillips screwdriver, remove the eight screws (F) holding the cover to the siderail and remove the cover.
5. Remove the yellow locking lever (G).
6. Using a 3/16" Allen key, remove the four Allen screws (H) holding each pommel to the siderail arms. Leave one loosened screw on each pommel until ready to remove both pommels.

#### NOTE

Apply medium-strength thread locker on the screw threads before replacing the screws.

7. While supporting the rail, remove the two last screws, the pommels, the bungee cord and finally the rail.

#### NOTE

Note the position of the pommel with the lock when the siderail is fully raised. The rail will not lock in high position if this pommel is positioned differently.

8. Reverse the above steps to install the new rail.
9. Test the siderail for proper operation before returning the bed to service.

## HEAD END RAIL REPLACEMENT

Please contact the Technical Service (see section 1.2) for information regarding the replacement of a head end rail.

**SIDERAIL MECHANISM TRANSFER PLATE REPLACEMENT****Required Tools:**

1/2" Wrench

**Procedure:****NOTE**

Unless otherwise indicated, refer to figure 4.1B on page 19 for the illustration of this procedure reference points.

1. Run the bed fully up and apply the brakes. Raise the siderail needing repair.
2. Unplug the power cord from the wall receptacle.
3. Remove the spring (G).
4. Using a 1/2" wrench, remove the locknut/shoulder spacers/bolt (H) holding each end of the transfer plate to the mechanism and remove the plate.
5. Reverse the above steps to install the new transfer plate.
6. Test the siderail for proper operation before returning the bed to service.

**FOOT END SIDERAIL MECHANISM REPLACEMENT****Required Tools:**

Small Slotted Head screwdriver  
3/16" Allen Key

Bungee Cord  
1/2" Socket Wrench

Phillips screwdriver  
Torque Wrench

**Procedure:**

1. Follow steps 1 to 7 of the foot end rail replacement procedure on page 20.
2. Using a 1/2" socket wrench, remove the four thread-rolling bolts (K, fig. 4.1B, page 19) holding the mechanism assembly to the support and remove the assembly.

**NOTE**

Once removed, the thread-rolling bolts must be first be screwed in manually before tightening them to a 130 lbf in (14.7 N m) torque using a torque wrench.

3. Reverse the above steps to install the new mechanism assembly.
4. Test the siderail for proper operation before returning the bed to service.

**HEAD END SIDERAIL MECHANISM REPLACEMENT**


Please contact the Technical Service (see section 1.2) for information regarding the replacement of a head end siderail mechanism.

## SIDERAIL CONTROL MEMBRANE REPLACEMENT

### Required Tools:

Small Slotted Head Screwdriver

### Procedure:

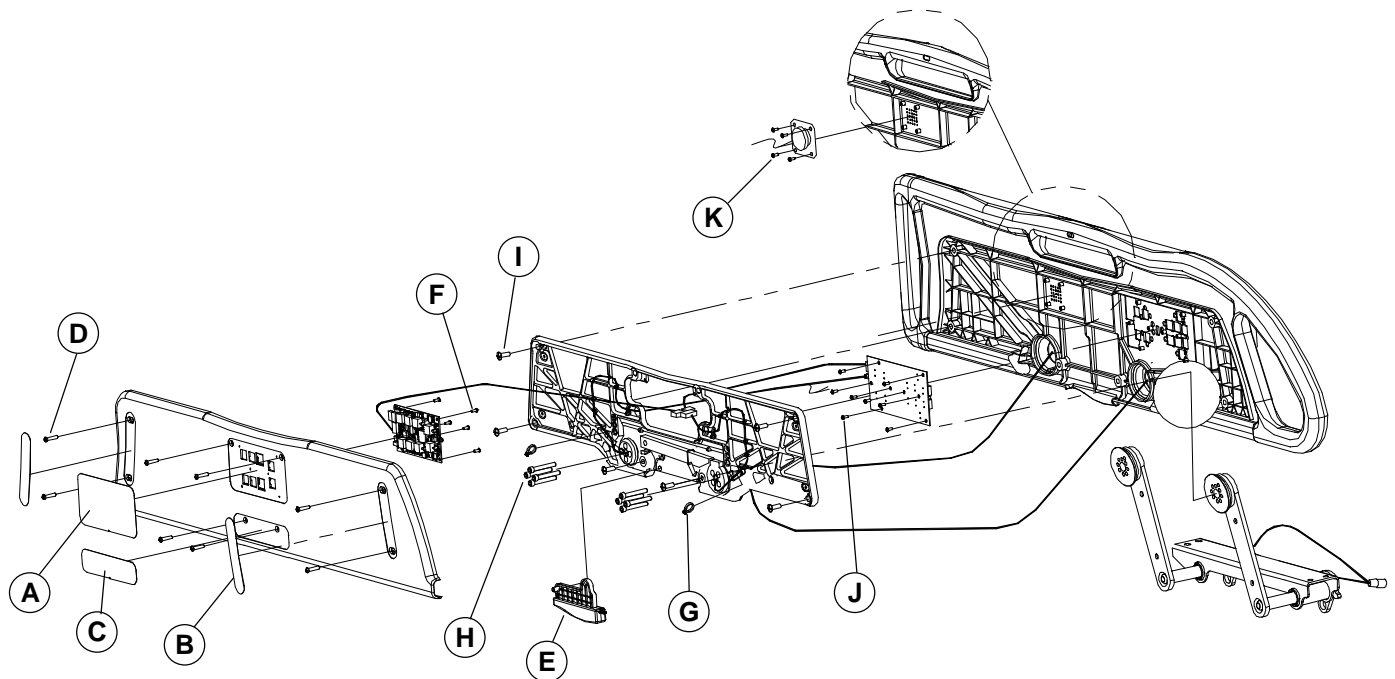

**Figure 4.1C**

1. Run the bed fully up and apply the brakes. Raise the siderail needing repair.
2. Unplug the bed power cord from the wall outlet.
3. Using a small slotted head screwdriver, lift and remove the self-sticking membrane (A) of the inner or outer control panel needing to be replaced. Proceed gently when inserting the screwdriver under the membrane to avoid scratching the molded rail.
4. Install the new membrane. Be sure to position the membrane properly over the board switches.
5. Test all controls of the control panel for proper operation before returning the bed to service.

## SIDERAIL OUTER PANEL CONTROL BOARD REPLACEMENT

### Required Tools:

Small Slotted Head Screwdriver

Phillips Screwdriver

### Procedure:

#### NOTE

Unless otherwise indicated, refer to figure 4.1C above for the illustration of this procedure reference points.

1. Run the bed fully up and apply the brakes. Raise the siderail needing repair.
2. Unplug the bed power cord from the wall outlet.
3. Using a small slotted head screwdriver, lift and remove the following self-sticking parts: the two screw-covers (B), the "Lift to release siderail" label (C) and the control panel membrane (A) to expose the screws holding the siderail cover. Proceed gently when inserting the screwdriver under the labels to avoid scratching the molded rail.

**NOTE**

Do not reuse the self-sticking parts removed since their self-adhesive coating considerably loses its efficiency once they are removed. We recommend that you have replacement stickers at hand (see Recommended Spare Parts on page 13).

4. Properly ground yourself (see section 1.4).
5. Using a Phillips screwdriver, remove the eight screws (D) holding the cover to the siderail. Remove the cover after having disconnected the cable of the control board attached to the cover.

**NOTE**

Gather the cables and place them properly before replacing the cover.

6. Remove the yellow locking lever (E).
7. Using a Phillips screwdriver, remove the six screws (F) holding the control board to the cover and remove the control board. Remove the cable connected to the board.
8. Reverse the above steps to install the new control board.
9. Test all controls of the outer control panel for proper operation before returning the bed to service.

**SIDERAIL INNER PANEL CONTROL BOARD REPLACEMENT****Required Tools:**

Small Slotted Head Screwdriver  
Bungee Cord  
Medium-Strength Thread Locker

Phillips Screwdriver  
3/16" Allen Key

Side Cutters

**Procedure:****NOTE**

Unless otherwise indicated, refer to figure 4.1C on page 22 for the illustration of this procedure reference points.



1. Follow steps 1 to 6 of the siderail outer panel control board replacement procedure above.
2. Secure the siderail needing repair to the adjacent siderail using a bungee cord (see opposite illustration).
3. Using side cutters, clip the lower cable tie (G) holding the bottom part of each grey cable to the aluminum structure. This loose will be necessary for steps to come.

**NOTE**

Make sure the two cable ties are inserted in the aluminum structure **before** attaching the structure.

4. Using a 3/16" Allen key, remove the four Allen screws (E) holding each pommel to the siderail arms. Leave one loosened screw on each pommel until ready to remove both pommels.

**NOTE**

Apply medium-strength thread locker on the screw threads before replacing the screws.



5. While supporting the rail, remove the two last screws and the bungee cord.
6. Lift the siderail, pass it over the mechanism arms and lay it on the mattress support while gently pulling on the cables (see opposite illustration).
7. Using a Phillips screwdriver, remove the six screws (I) holding the aluminum structure to the siderail.
8. Grasp and lift the upper part of the aluminum structure lying on the mattress support until it may be secure temporarily to one of the mechanism arms using a cable tie.
9. Remove from the defective board the cable connected to it. The optional nurse call cable may also be present, remove it.
10. Using a Phillips screwdriver, remove the eight screws (J) holding the control board to the siderail.
11. Reverse the above steps to install the new control board.
12. Test all controls of the inner/outer control panels for proper operation before returning the bed to service.

## **SIDERAIL INNER PANEL SPEAKERPHONE REPLACEMENT**

### **Required Tools:**

Small Slotted Head Screwdriver	Phillips Screwdriver	Side Cutters
Bungee Cord	3/16" Allen Key	
Soldering Iron and Soft Welding Wire		

### **Procedure:**

1. Follow steps 1 to 8 of the siderail inner panel control board replacement procedure, described above.
2. Using a Phillips screwdriver, remove the four screws (K, fig. 4.1C, page 22) holding the speakerphone to the siderail.
3. Using a soldering iron, unsolder the two wires attached to the defective speakerphone and remove the speakerphone. Do the same with the wires attached to the new speakerphone. Keep the wires as replacement parts.
4. Using a soldering iron and soft welding wire, attach the two wires to the new speakerphone. Clean the welds.

### **NOTE**


Before reassembling the different parts of the siderail, we recommend that you check the continuity at the speakerphone terminals to ensure the circuit operates properly.

5. Reverse the preceding steps to install the new speakerphone.
6. Test the nurse call and all controls of the inner/outer control panels for proper operation before returning the bed to service.



**4.2 FOOT BOARD MAINTENANCE PROCEDURES****FOOT END CONTROL MEMBRANE REPLACEMENT****Required Tools:**

Small Slotted Head Screwdriver

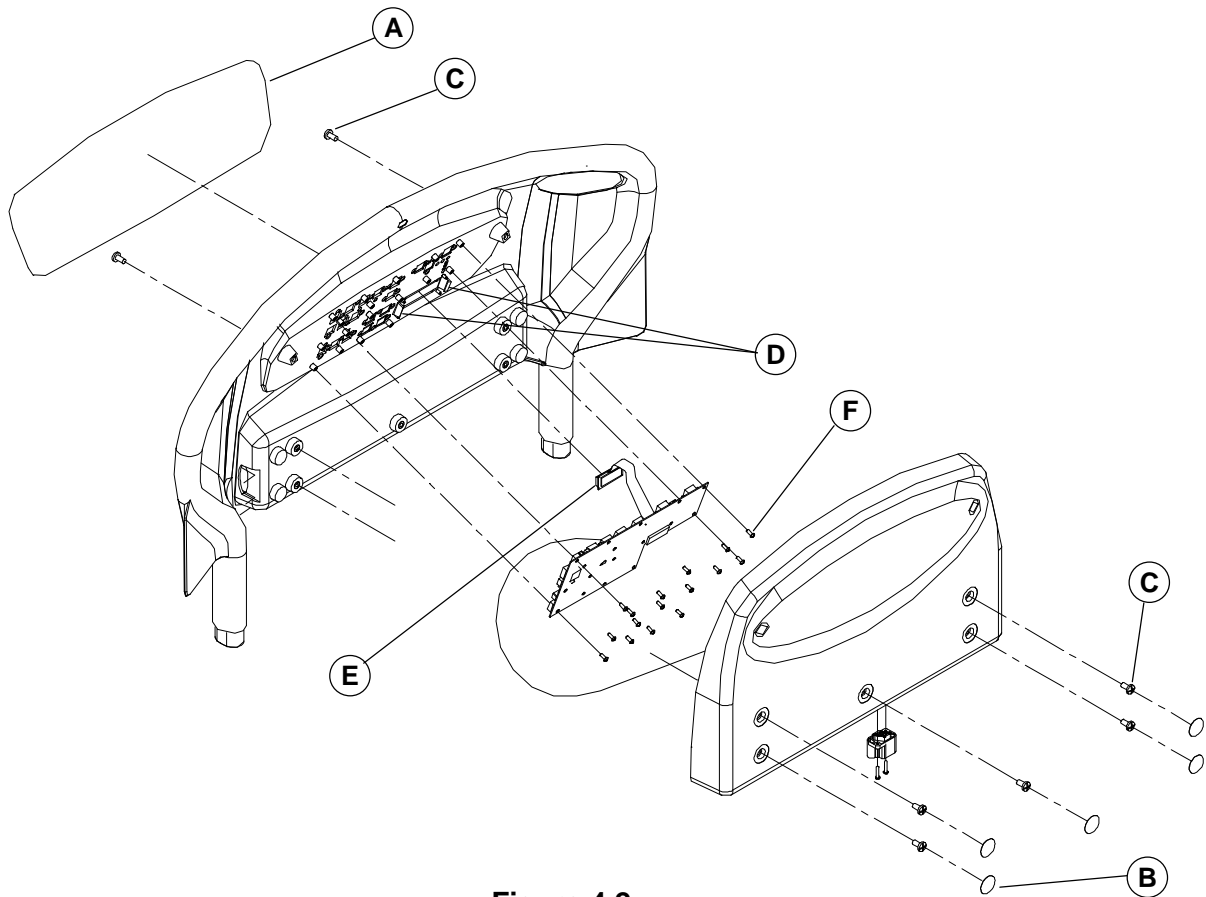**Procedure:****Figure 4.2**

1. Run the bed fully up and apply the brakes.
2. Unplug the bed power cord from the wall outlet.
3. Using a small slotted head screwdriver, lift and remove the self-sticking membrane (A). Proceed gently when inserting the screwdriver under the membrane to avoid scratching the molded rail.
4. Install the new membrane. Be sure to position the membrane properly over the board switches.
5. Test all controls of the foot board control panel for proper operation before returning the bed to service.

## **WEIGH SYSTEM LCD DISPLAY (OPTIONAL) REPLACEMENT**

### **Required Tools:**

Small Slotted Head Screwdriver

Phillips Screwdriver

### **Procedure:**

#### **NOTE**

Unless otherwise indicated, refer to figure 4.2 on page 25 for the illustration of this procedure reference points.

1. Run the bed fully up and apply the brakes.
2. Unplug the bed power cord from the wall outlet.
3. Using a small slotted head screwdriver, lift and remove the five self-sticking round screw-covers (B) located on the foot board cover as well as the self-sticking membrane (A) of the foot board control panel to expose the screws holding the cover to the foot board.

#### **NOTE**

Do not reuse the self-sticking parts removed since their self-adhesive coating considerably loses its efficiency once they are removed. We recommend that you have replacement stickers at hand (see Recommended Spare Parts on page 13).

4. Properly ground yourself (see section 1.4).
5. Using a Phillips screwdriver, remove the seven screws (C) holding the cover to the foot board and remove the cover after having disconnected the cable of the foot board interface plug from the control board.
6. Using a small slotted head screwdriver, push aside the clips (D) and remove the LCD display (E) after having disconnected its cable from the control board.
7. Reverse the above steps to install the new LCD display.
8. Test the LCD display for proper operation before returning the bed to service.

## **FOOT END CONTROL BOARD REPLACEMENT**

### **Required Tools:**

Small Slotted Head Screwdriver

Phillips Screwdriver

### **Procedure:**


#### **NOTE**

Unless otherwise indicated, refer to figure 4.2 on page 25 for the illustration of this procedure reference points.

1. Follow steps 1 to 5 of the weigh system LCD display replacement procedure above.
2. Disconnect all cables from the control board. Note the locations of the cables so they will be reconnected properly to the new control board.
3. Using a Phillips screwdriver, remove the 17 screws (F) holding the control board to the foot board and remove the control board.
4. If need be, remove the weigh system ACL display (optional) (E) by pushing aside the clips (D) using a small slotted head screwdriver. Disconnect its cable from the board and install the display on the new control board.
5. Reverse the above steps to install the new control board.
6. Test all controls of the foot board control panel for proper operation before returning the bed to service.

**4.3 MATTRESS SUPPORT MAINTENANCE PROCEDURES****FOOT SECTION REPLACEMENT****Required Tools:**

1/2" Wrench

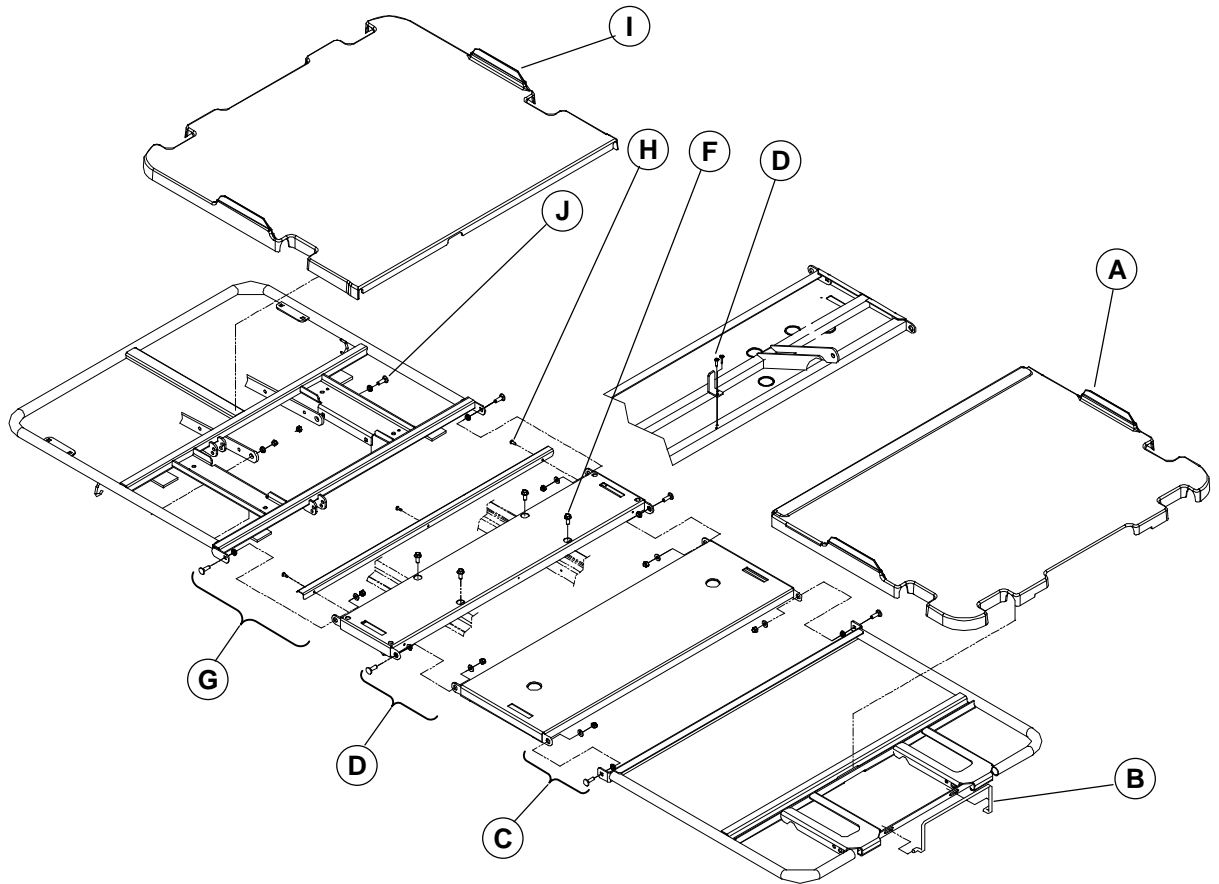**Procedure:**

**Figure 4.3A**

1. Run the bed fully up and apply the brakes. Flatten all sections of the mattress support.
2. Unplug the bed power cord from the wall outlet.
3. Lower the siderails.
4. Remove the foot section molded shell (A).
5. Lift and fold back the foot section toward the head end of the bed.
6. Remove the mattress retainer (B) and install it on the new foot section. Note its position before removing it. Replace the foot section to horizontal position.
7. Using a 1/2" wrench, remove the two locknuts/washers/shoulder spacers/bolts (C) linking the foot section to the thigh section and remove the defective section.
8. Reverse the above steps to install the new foot section.

## THIGH SECTION REPLACEMENT

### Required Tools:

Long Nose Pliers

1/2" wrench

Phillips Screwdriver

### Procedure:

#### NOTE

Unless otherwise indicated, refer to figure 4.3A for the illustration of this procedure reference points.

1. Run the bed fully up and apply the brakes. Flatten all sections of the mattress support.
2. Unplug the bed power cord from the wall outlet.
3. Lower the siderails.
4. Remove the foot and head section molded shells.
5. Using long nose pliers, remove the rue ring cotter/washers/clevis pin (F, fig. 4.3B, page 30) linking the thigh section lever arms to the thigh actuator tube.
6. Using a 1/2" wrench, remove the four locknuts/washers/shoulder spacers/bolts (C, D) linking the thigh section to the foot and seat sections.
7. Remove the defective thigh section and lay it upside down on a workbench.
8. Using a Phillips screwdriver, remove the two screws (E) holding the micro switch activator to the thigh section. Install the activator on the new thigh section.
9. Reverse the above steps to install the new thigh section.
10. Test the Knee Gatch as well as the Auto Contour and cardiac chair before returning the bed to service.

## SEAT SECTION REPLACEMENT

### Required Tools:

1/2" Long Socket Wrench

1/2" Wrench

Phillips Screwdriver

### Procedure:

#### NOTE

Unless otherwise indicated, refer to figure 4.3A for the illustration of this procedure reference points.

1. Raise the bed fully up and apply the brakes. Flatten the mattress support.
2. Unplug the power plug from the wall outlet.
3. Lower the siderails.
4. Using a 1/2" long socket wrench, remove the four screws (F) holding the seat section to the frame.
5. Using a 1/2" wrench, remove the four locknuts/washers/shoulder spacers/bolts (D, G) linking the seat section to the thigh and head section and remove the defective seat section.
6. Using a Phillips screwdriver, remove the three screws (H) holding the protective plate to the seat section.
7. Reverse the preceding steps to install the new seat section.

**HEAD SECTION REPLACEMENT****Required Tools:**

1/2" Socket wrench

1/2" Wrench (2)

Side Cutters

**Procedure:****NOTE**

Unless otherwise indicated, refer to figure 4.3A for the illustration of this procedure reference points.

1. Raise the bed fully up and apply the brakes.
2. Unplug the bed power cord from the wall outlet.
3. Remove the molded shell (I) of the head section and raise the head siderails.
4. Loosen the lock rings (A, fig. 4.1A, page 18) and remove the siderail control cables.
5. Using a 1/2" socket wrench, remove the eight thread-rolling bolts (B, fig. 4.1A, page 18) holding the head siderail assemblies to the head section and remove the siderails. Support the assembly when removing the last bolts.
6. Using two 1/2" wrenches, remove the locknut/bolt (M, fig. 4.3H, page 45) holding the cylinder end fitting to the head section.

**NOTE**

Note the use of the lever lower holes to attach the cylinder end fitting.

7. Using side cutters, remove the cable tie holding the cables on each side of the head section structure.
8. Using two 1/2" wrenches, remove the nut/shoulder spacers/bolt (N, fig. 4.3H, page 45) holding the upper part of the head section lever to the head section structure.

**NOTE**

Note the use of the lever upper holes to attach the head section lever.

9. Using a 1/2" wrench, remove the locknut/shoulder spacers/bolt (J) holding the upper part of each head section arm to the head section structure.
10. Using a 1/2" wrench, remove the two locknuts/bolts (L, fig. 4.3H, page 45) holding each CPR release handle to the head section structure.
11. Using a 1/2" wrench, remove the two locknuts/shoulder spacers/washers/bolts (G) holding the head section to the seat section.
12. Reverse the above steps to install the new head section.
13. Test the CPR mechanism for proper operation before returning the bed to service.

## FOWLER ACTUATOR REPLACEMENT

### Required Tools:

Long Nose Pliers

Phillips Screwdriver

Pliers

Side Cutters

### Procedure:

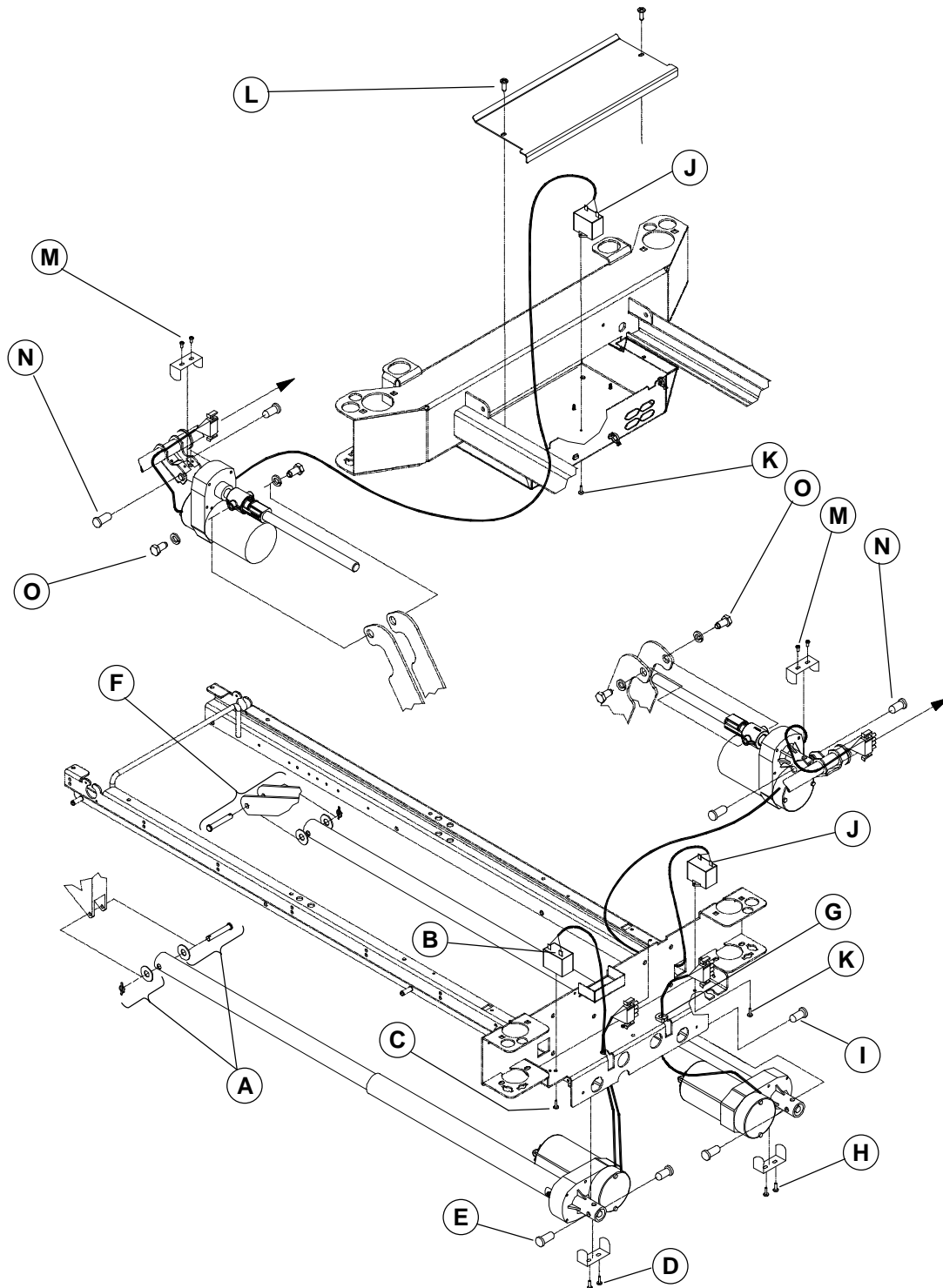


Figure 4.3B

1. Raise the bed fully up and apply the brakes. Raise the Knee Gatch fully up.
2. Unplug the bed power cord from the wall outlet.
3. Lower the siderails.

4. Remove the molded shells from the foot and head sections.
5. Lift and fold the foot section back toward the head end of the bed.
6. Using long nose pliers, remove the rue ring cotter/washers/clevis pin (A) linking the actuator tube to the head section lever arms.
7. Remove the head board.
8. Using a Phillips screwdriver, remove the six screws (A, fig. 4.3D, page 36) holding the cover to the foot end casing. Grab both ends of the cover and lift up to remove it.
9. Properly ground yourself (see section 1.4).
10. Using side cutters, clip the cable tie holding the head actuator cable to the other cables.
11. Remove the actuator cable (connector J8) from the control board and pass the cable connector through the rear casing hole.
12. Remove from the capacitor the two wires (B) connected to it and remove, using a Phillips screwdriver, the screw (C) holding the capacitor to the foot casing. Dispose of the capacitor.
13. Using a Phillips screwdriver, remove the two screws (D) holding the retaining plate to the actuator support.
14. Remove the two pivot pins (E) holding the actuator to the support. To facilitate the removal of the pins, insert a small slotted head screwdriver into the opening at the end of the actuator and push out the pins.
15. Move the actuator toward the centre of the bed to remove it from its location.
16. Reverse the above steps to install the new actuator. Take note of the following caution before hooking up the actuator tube to the head section lever arms.



---

**CAUTION**

**It is of utmost importance** that the course of the new head actuator be adjusted before hooking up its tube to the head section lever arms. An improper adjustment can damage the head section structure.

17. To adjust the course of the replacement head actuator, proceed as follows:
  - A. Make sure that the actuator cable is connected to the control board and connect the bed power cord.
  - B. Grab the new actuator tube and position its holes horizontally. While holding the tube firmly to prevent it from rotating, press the Fowler up control for a few seconds, then press the down control until the actuator stops. This will be the lower limit of the actuator course.
  - C. Gently turn the tube in either direction to align the tube holes with those of the head section lever arms. Then raise again the Fowler a few inches while holding firmly the tube and lower it completely.
  - D. Check the alignment of the holes. If the tube holes are not aligned any more with those of the lever arms, repeat step **C** until they are. Once the holes aligned, install the washers and the clevis pin. Slightly raise the head section manually to ease the insertion of the clevis pin.
  - E. Before installing the rue ring cotter to finalize the head actuator replacement, raise and lower completely the Fowler. Check that the actuator stops working as soon as the head section reaches the frame.

## KNEE GATCH ACTUATOR REPLACEMENT

### Required Tools:

Long Nose Pliers  
Side Cutters

Phillips Screwdriver

Pliers

### Procedure:

#### NOTE

Unless otherwise indicated, refer to figure 4.3B on page 30 for the illustration of this procedure reference points.

1. Raise the bed fully up and apply the brakes. Raise the Knee Gatch fully up.
2. Unplug the bed power cord from the wall outlet.
3. Lower the siderails.
4. Remove the molded shells from the foot and head section.
5. Lift and fold the foot section back toward the head end of the bed.
6. Using long nose pliers, remove the rue ring cotter/washers/clevis pin (F) linking the actuator tube to the thigh section lever arms.
7. Using a Phillips screwdriver, remove the six screws (A, fig. 4.3D, page 36) holding the cover to the foot end casing. Grab both ends of the cover and lift up to remove it.
8. Properly ground yourself (see section 1.4).
9. Using side cutters, clip the cable tie holding the thigh actuator cable to the other cables.
10. Remove the actuator cable (connector J9) from the control board.
11. Using pliers, squeeze the upper part of the strain-relief bushing (G) and lift up remove it from its location.
12. Remove the cable from the bushing and pass the cable connector through the casing hole previously occupied by the bushing.
13. Using a Phillips screwdriver, remove the two screws (H) holding the retaining plate to the actuator support.
14. Remove the two pivot pins (I) holding the actuator to the support. To facilitate the removal of the pins, insert a small slotted head screwdriver into the opening at the end of the actuator and push out the pins.
17. Move the actuator toward the centre of the bed to remove it from its location.
18. Reverse the above steps to install the new actuator. Take note of the following caution before hooking up the actuator tube to the head section lever arms.



### CAUTION

**It is of utmost importance** that the course of the new thigh actuator be adjusted before hooking up its tube to the thigh section lever arms. An improper adjustment can damage the thigh section structure.

19. To adjust the course of the replacement thigh actuator, proceed as follows:
  - A. Make sure that the actuator cable is connected to the control board and connect the bed power cord.
  - B. Grab the new actuator tube and position its holes horizontally. While holding the tube firmly to prevent it from rotating, press the Knee Gatch up control for a few seconds, then press the down control until the actuator stops. This will be the lower limit of the actuator course.
  - C. Gently turn the tube in either direction to align the tube holes with those of the thigh section lever arms. Then raise again the Knee Gatch a few inches while holding firmly the tube and lower it completely.



- D. Check the alignment of the holes. If the tube holes are not aligned any more with those of the lever arms, repeat step **C** until they are. Once the holes aligned, install the washers and the clevis pin. Slightly raise the thigh section manually to ease the insertion of the clevis pin.
- E. Before installing the pin ring cotter to finalize the actuator replacement, raise and lower completely the Knee Gatch. Check that the actuator stops working as soon as the thigh section reaches the frame.

## HI-LO ACTUATOR REPLACEMENT

### NOTE

In order to preserve the adjustment of the bed lowest position when replacing a Hi-lo actuator, a special tool kit designed for that purpose must be used. The kit includes alignment jigs. To obtain this kit, contact our Service department (see section 1.2) and order part number KR0113.

### Required Tools:

Tool Kit (KR0090)

Side Cutters

5/16" Socket Wrench

Angle Indicator

### Procedure:


### NOTE

Unless otherwise indicated, refer to figure 4.3B on page 30 for the illustration of this procedure reference points.

1. Position the mattress support sections depending on the location of the Hi-Lo actuator to replace:

**At the foot of the bed:** Remove the molded shell of the foot section, fully raise the thigh section, and lift and fold back the foot section toward the head end of the bed. Finally, using an angle indicator, raise the Fowler by approximately 30°.

**At the head of the bed:** Remove the molded shell of the head section and fully raise the Fowler.

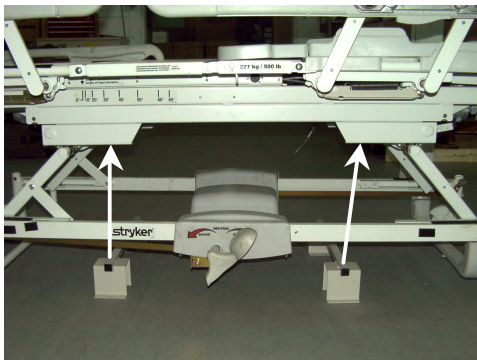

2. Position the alignment jigs on the floor right under the Hi-Lo levers and lower the bed until the levers come to rest on the jigs (see opposite illustration). Use a socket wrench with the 1/2" socket provided in the kit to lower a defective Hi-Lo actuator until the lever rests on the jig.
3. Unplug the bed power cord from the wall outlet.
4. Disconnect the actuator cable and clip, using side cutters, the cable ties holding it to the frame.
5. Remove from the capacitor the two wires (J) connected to it and remove, using a Phillips screwdriver, the screw (K) holding in place the capacitor. Dispose of the capacitor.
  - The capacitor of the head Hi-Lo actuator is located in the scale control board casing. To access it, remove, using a Phillips screwdriver, the two screws (L) holding it the cover plate.
  - The capacitor of the foot Hi-Lo actuator is located in the foot casing. To access it, remove the six screws (A, Fig. 4.3C, page 35) holding the cover to the foot casing.
6. Using a 5/16" socket wrench, remove the two screws (M) holding the retaining plate to the actuator support.
7. Remove the two pivot pins (J) holding the actuator to the support. To facilitate the removal of the pins, insert a small slotted head screwdriver into the opening at the end of the actuator and push out the pins.

8. Using a 1/2" wrench, remove the two bolts/washers/shoulder spacers (N) holding the molded nut support to the Hi-Lo lever.

**NOTE**

Make sure that the support and the molded nut holes are aligned before screwing in the bolts. If resistance is felt, it means that the holes are not aligned.

9. Remove the defective actuator.
10. Reverse the above steps to install the new actuator. Take note of the following caution before hooking up the actuator to the Hi-Lo lever.



**CAUTION**

The course of the new actuator must be adjusted prior to hooking it to the Hi-Lo lever. An improper adjustment can damage the Hi-Lo mechanism.

10. To adjust the new actuator, process as follows:
  - A. Once the new actuator cable connected, plug the bed power cord and press the bed down control until the actuator stops. This will be the lower limit of the actuator course.
  - B. Link the actuator to the Hi-Lo lever.
  - C. The alignment jigs still in position, raise and lower completely the bed to ensure that the lower limit is properly adjusted.

## MOTOR CONTROL BOARD REPLACEMENT

### Required Tools:


Phillips Screwdriver

Long-Nose Pliers

Side Cutters

3/8" Wrench

### Procedure:

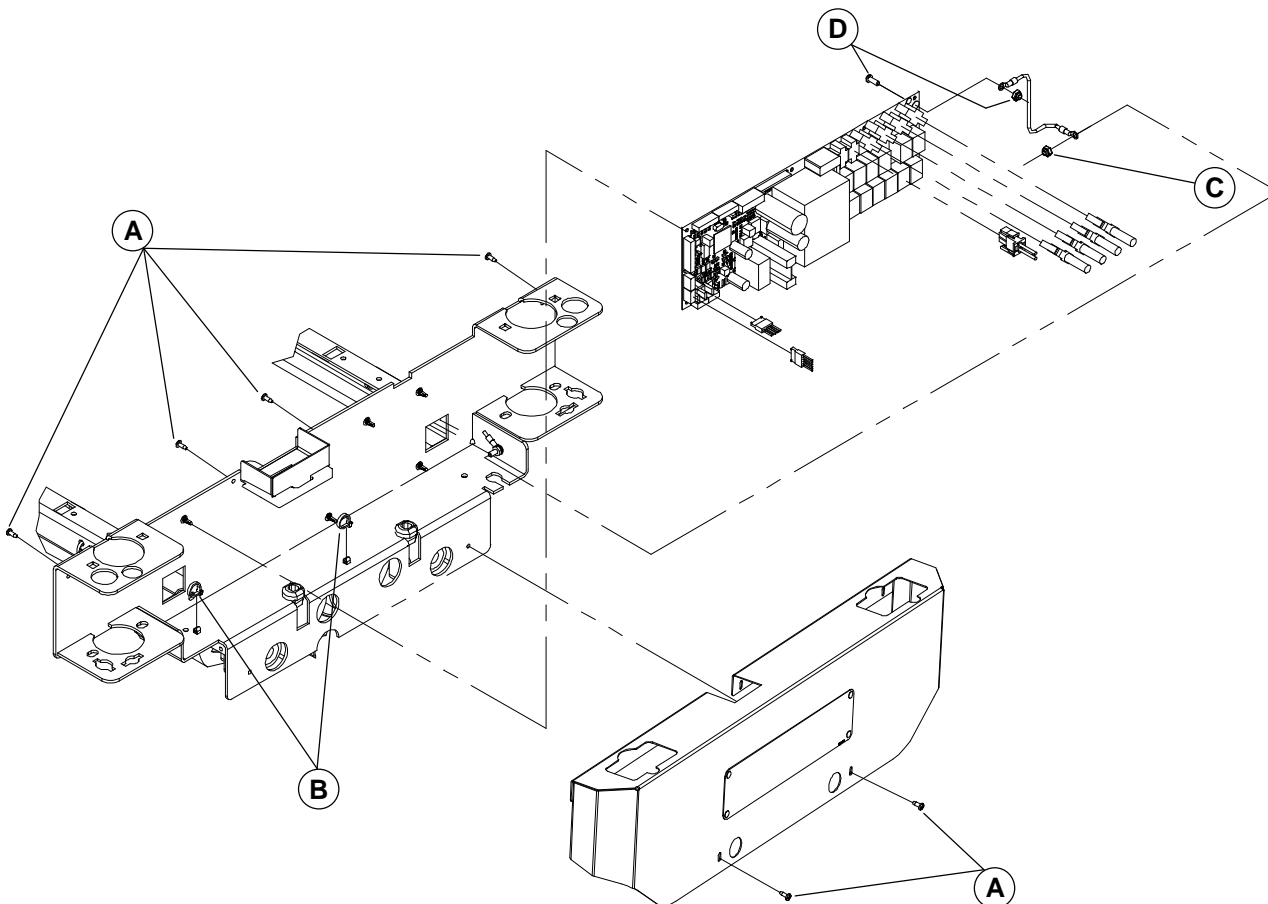

**Figure 4.3C**

1. Raise the bed fully up and apply the brakes.
2. Unplug the bed power cord from the wall outlet.
3. Remove the foot board.
4. Using a Phillips screwdriver, remove the six screws (A) holding the cover to the foot end casing. Grab both ends of the cover and lift up to remove it.
5. Properly ground yourself (see section 1.4).
6. Using side cutters, clip the two cable ties (B) holding together the cables.

#### NOTE

Carefully note the position and the gathering of the cables before clipping the cable ties.

7. Remove all cables connected to the control board. Note their location so they will be connected properly to the new board. Refer to drawing OL205024 for the connecting position of the cables on the motor control board

#### NOTE

Connector J2 of the control board receives the following cable connectors:

- Auto Contour (on pins 4-7) and Cardiac Chair (on pins 10-11);
- CPR (on pins 8-9).


8. Using a 3/8" wrench, remove the nut (C) holding the ground cable to the frame.
9. Using long-nose pliers, pinch the upper part of the stand-off pins and gently lift the board up and out.
10. Using a 3/8" wrench and a Phillips screwdriver, remove the nut/screw (D) holding the ground cable to the board.
11. Reverse the above steps to install the new control board.
12. Test all the bed functions before returning the bed to service.

## NURSE CALL/COMMUNICATIONS PACKAGE CONTROL BOARD (OPTIONAL) REPLACEMENT


### Required tools:

Phillips Screwdriver

### Procedure:

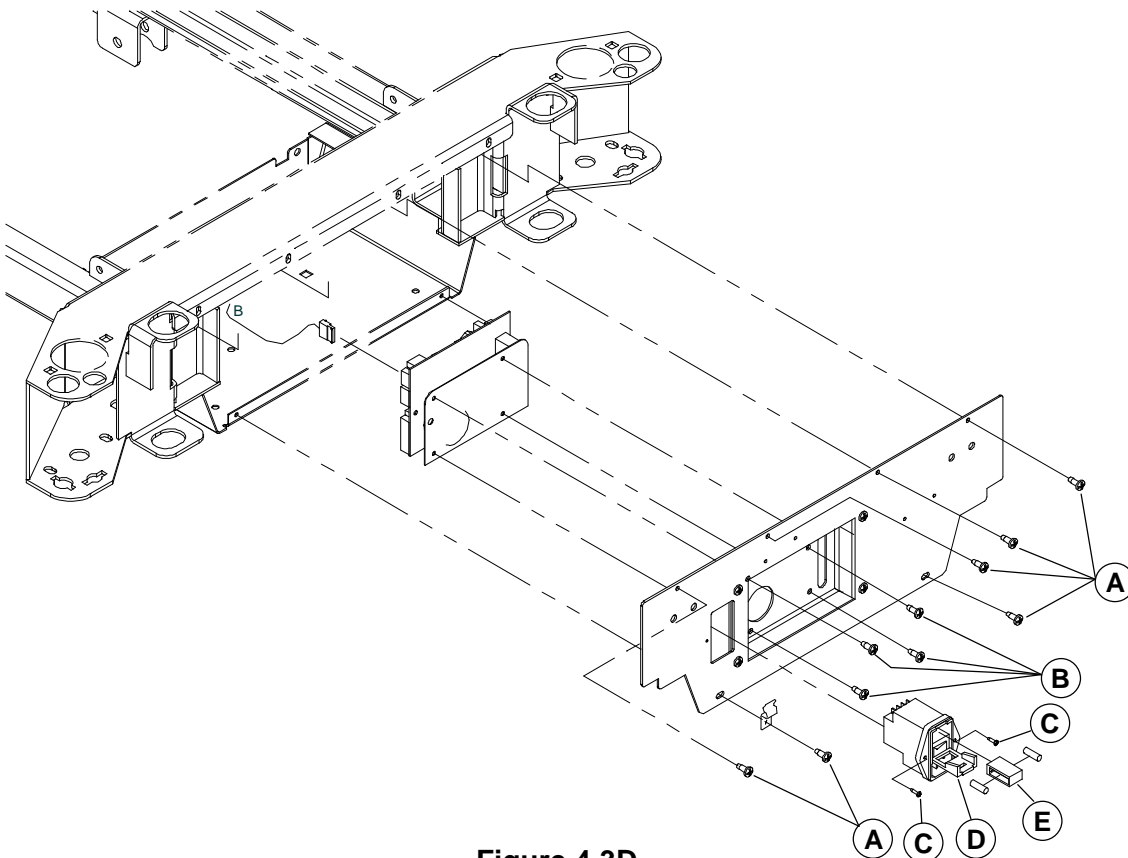

**Figure 4.3D**

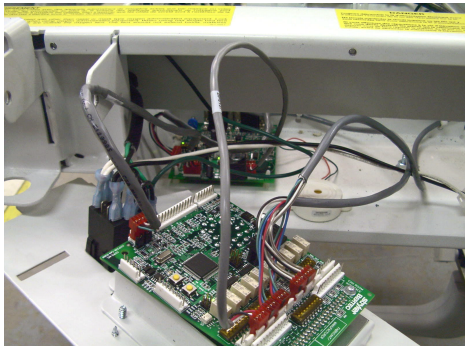

1. Raise the bed fully up and apply the brakes.
2. Unplug the bed power cord from the wall outlet.
3. Remove the power cord from the power connector.
4. Using a Phillips screwdriver, remove the six screws (A) holding the cover to the head casing.
5. Properly ground yourself (see section 1.4).
6. Once the screws removed, pivot the cover and lay it flat on the bottom of the casing (see opposite illustration).

7. Remove all cables connected to the control board. Note their location so they will be connected properly to the new board.
8. Pivot the cover to vertical position and, using a Phillips screwdriver, remove the four screws (B) holding the control board to the head casing cover and remove the board.
9. Reverse the above steps to install the new control board.
10. Test the nurse call (optional) for proper operation before returning the bed to service.

## **POWER CONNECTOR REPLACEMENT**

### **Required Tools**

Phillips Screwdriver

### **Procedure:**

#### **NOTE**

Unless otherwise indicated, refer to figure 4.3B on page 30 for the illustration of this procedure reference points.

1. Follow steps 1 to 6 of the nurse call/communications package control board replacement procedure described above.
2. Remove all cables connected to the power connector. Note their location so they will be connected correctly to the new power connector. Refer to drawing OL250024 for the connecting position of the cables on the power connector.
3. Pivot the cover to vertical position and remove, using a Phillips screwdriver, the two screws (C) holding the power connector to the cover.
4. Press the connector clips and remove it from its location.
5. Reverse the above steps to install the new power connector.
6. Test that the bed is powered before returning it to service.

## **POWER CONNECTOR FUSE REPLACEMENT**

### **Required Tools**

Small Slotted Head Screwdriver

### **Procedure:**

#### **NOTE**

Unless otherwise indicated, refer to figure 4.3B on page 30 for the illustration of this procedure reference points.

1. Remove the power cord from the power connector.
2. Using a small slotted screwdriver, open and slide down the power connector door (D).
3. Still using the small screwdriver, remove the fuse holder (E).
4. Remove the defective fuse and replace it by a new one.

#### **NOTE**

Fuses used in the MA204 bed, powered by the 120V electrical system, are of the fast acting type and their rating is 250V, 10A. For beds powered by other electrical system, see section 1.1 "Specifications" for fuse ratings.

5. Replace the fuse holder in its housing and close the door.

#### **NOTE**

There is only one way to install the fuse holder. The characters "250V" must be apparent through the door small window. If replaced the wrong way, the bed will not be powered.

6. Test the power switch for proper operation before returning the bed to service.

## SCALE CONTROL BOARD/BED EXIT BUZZER (OPTIONAL) REPLACEMENT


### Required Tools:

Phillips Screwdriver

3/8" Socket Wrench

Long-Nose Pliers

### Procedure:

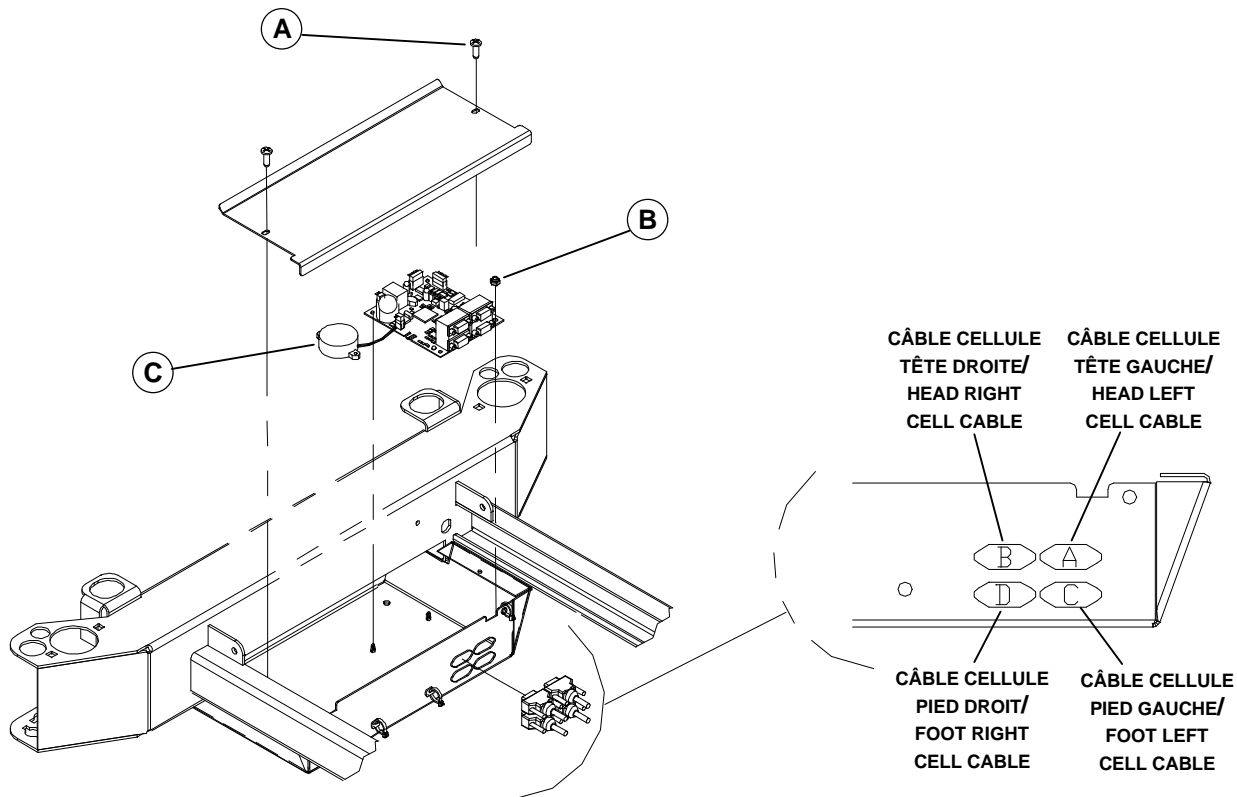


**Figure 4.3E**

1. Raise the bed fully up and apply the brakes.
  2. Raise the Fowler fully up.
  3. Unplug the bed power cord from the wall outlet.
  4. Using a Phillips screwdriver, remove the two screws (A) holding the cover plate.
  5. Properly ground yourself (see section 1.4).
- To replace the Bed Exit Buzzer, proceed with steps 7 to 10
- To replace the scale control board, proceed with steps 11 to 12
6. Disconnect from the control board the buzzer cable.
  7. Lift the buzzer (C) to remove it from the plate. Clean any glue residue left.
  8. Reverse the above steps to install the new buzzer.
  9. Test the Bed exit system to make sure the buzzer operates properly.
  10. Remove all cables connected to the control board. Note their location so they will be connected correctly to the new power connector. Refer to OL250041 for the connecting position of the cables on the scale control board.
  11. Using a 3/8" socket wrench, remove the ground nut (B).
  12. Using long-nose pliers, pinch the upper part of the stand-off pins and gently lift the board up and out.
  13. Reverse the above steps to install the new control board.
  14. Calibrate the scale before returning the bed to service. Refer to the scale calibration procedure on page 42.

**TOROIDAL TRANSFORMER REPLACEMENT (INTERNATIONAL SERIES BEDS)****Required Tools:**

Tournevis Phillips

1/2" Wrench

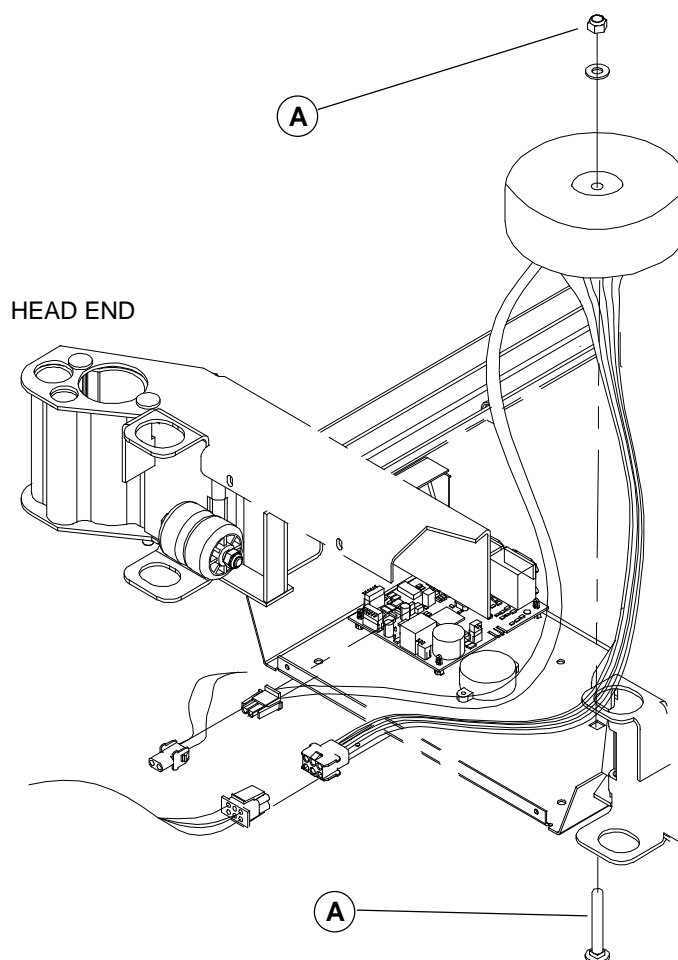**Procedure:****Figure 4.3K**

1. Raise the bed fully up and apply the brakes.
2. Raise the Fowler fully up.
3. Unplug the bed power cord from the wall socket.
4. Using a Phillips screwdriver, remove the two screw (A, fig. 4.3E, page 38) holding the cover to the casing.
5. Properly ground yourself (see section 1.4).
6. Using a Phillips screwdriver, remove the nut/washer/bolt (D, fig. 4.3E, page 38) holding the transformer to the casing. Disconnect the transformer cable and remove it.

**NOTE**

Tighten the bolt lightly but enough.

7. Reverse the above steps to install the new transformer.
8. On a bed equipped with the optional Weigh system, calibrate the scale (see page 42). If only the optional Bed Exit system is present on the bed, zero the system (see the section dealing with the Bed Exit (Chaperone) in the Operations Manual).
9. Test all the bed electric controls for proper operation before returning the bed to service.

## LOAD CELL (OPTIONAL) REPLACEMENT

### Required Tools:

Long-Nose Pliers

Side Cutters

Jack Stands (2)

1/4" Allen key

1/2" Wrench

### Procedure:



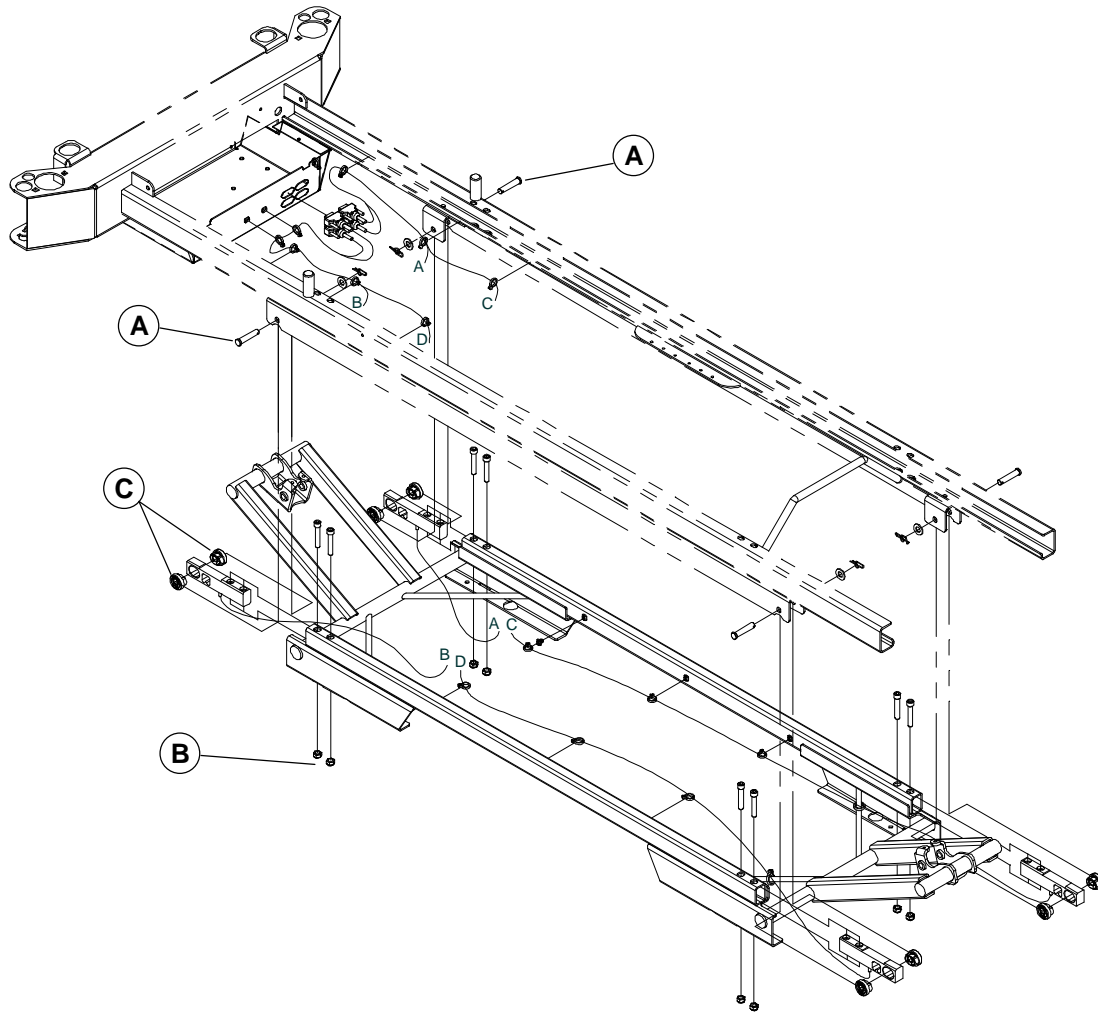


Figure 4.3F

1. Raise the bed fully up and apply the brakes.
2. Unplug the bed power cord from the wall outlet.
3. Depending on the location of the load cell to replace, proceed with the following operation:  
**Head end of the bed:** Remove the head board and the molded shell of the head section.  
**Foot end of the bed:** Remove the foot board and the molded shell of the foot section.
4. Using long-nose pliers, remove the two rue ring cotters/washers/clevis pins (A) holding the end of the defective load cell as well as the end of the one adjacent to it. It is important that the end of **both cells** be freed and their cables loosen for the coming steps.
5. Using side cutters, clip the two first cable ties holding the defective cell cable to the frame immediately after the cell. Do the same for the adjacent cell.
6. Plug in the bed power cord.

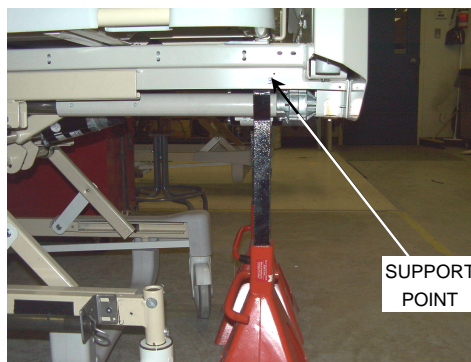

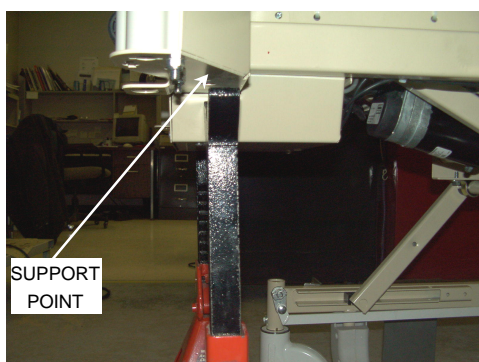


7. Place two jack stands under the frame at the head or foot end of the bed, depending on the location of the load cell to replace (see above illustration).
8. Lower the bed until the Allen screws holding the defective load cells to the frame are easily accessible.
9. Unplug the bed power cord from the wall outlet.
10. Using a 1/4" Allen key and a 1/2" wrench, remove the two nuts/Allen screws (B) holding the defective load cell to the frame.
11. Disengage the defective load cell cable from the cable ties holding it along its path to the scale control board and remove it from the board. Remove the defective cell.
12. Inspect the elastomer sleeves (C) to ensure that the inner circular hole has not become elliptic. Replace if it is the case.
13. Reverse the above steps to install the new load cell.
14. Proceed with the calibration of the scale before returning the bed to service. Refer to the scale calibration procedure on page 42.

### **SCALE (OPTIONAL) CALIBRATION**

#### **• Verifying the Scale Accuracy**

1. Empty the bed from all unnecessary material (accessories, linen, mattress, etc.) and zero the scale by doing the following:
  - Press  to activate the scale. The screen will display:
 

WEIGHT	ANGLE
XXX.X KG	+/-XX.X°
  - Press and hold ZERO. Display will read:
 

HOLD TO ZERO WT., followed by:  
 RELEASE TO ZERO (release ZERO), followed by:  
 DO NOT TOUCH BED (make sure nobody touches the bed while this message is displayed)

Once done with the zeroing, the system will return to the Scale mode and will display zero for the weight and the current angle value.
2. Place a calibrated weight on the center of the mattress support. The weight displayed should be within the following range:  $\pm 2\%$  for a weight equal or over 100 lb (45.4 kg) or  $\pm 2$  lb (0.9 kg) if the weight is lower than 100 lb (45.4 kg).
3. If the weight displayed is incorrect, remove the weight from the bed and calibrate the scale.

## • Scale Calibration



### **WARNING**

If the scale calibration procedure cannot be completed once it has been started, turn the main switch off and turn it back on to restore the previous calibration data

### **NOTE**

The scale calibration is done in three steps. Steps 2 and 3 come in automatically as soon as a step is over. Once the calibration process is over, the system returns to the main maintenance menu.

In the maintenance context, the ZERO key enables the return to the previous level in the hierarchy of the maintenance menu.

Note that the system will automatically shut off after one minute of idle time when you are at the main maintenance menu level. Should this happen, the whole process to access the main maintenance menu will have to be repeated.

### **Required Tools:**

Angle Meter

### **Procedure:**

1. Power off the bed using the main switch at the head end of the bed.
2. Power on the bed and reach the foot end of the bed to simultaneously press both Up/Down keys of the scale interface menu. This operation allows access to the main maintenance menu. Display will read:

► DEBOG MENU  
CALIB. MENU ▼

### **NOTE**

You have exactly five seconds to reach the foot end of the bed and press the keys after having powered the bed. The system will shut off after this delay, simply repeat the process should it happen.

3. Choose CALIB. MENU using the Down key and press ENTER. Display will then read:

PUT BED AT 0°  
PRESS ENTER

4. Put the angle meter on the seat section of the mattress support and position the bed to 0°. Then press ENTER. Display will read:

DO NOT TOUCH BED

5. As indicated, do not touch the bed until the end of this step. This is the longest of the three steps. Once over, the system will bring in the second step. Display will then read:

PUT BED AT +12°  
PRESS ENTER

6. Incline the bed in the Trendelenburg position (head down, foot up) until the meter reads +12°. Ensure that the same material that was on the bed in step 1 stays in the bed. Then press ENTER. Display will then read:

DO NOT TOUCH BED

7. As indicated, do not touch the bed until the end of this step. Once over, the system will automatically bring in the third and last step. Display will then read:

PUT BED AT -12°  
PRESS ENTER

8. Incline the bed in the Reverse Trendelenburg position (head up, foot down) until the meter reads -12°. Ensure that the same material that was on the bed in step 2 stays in the bed. Then press ENTER. Display will then read:

DO NOT TOUCH BED

9. As indicated, do not touch the bed until the end of this step. Once the last step is over, the system will automatically bring in the main maintenance menu.
10. To get back to the normal scale display, press ZERO.
11. Before returning the bed to service, Test the scale accuracy by following the operations described above under " Verifying Scale Accuracy".

**NOTE**


It is possible to leave the calibration process any time before step 3 is completed by pressing the ZERO key. The preceding calibration data will be restored.

If the bed is placed at the wrong angle in step two or three, the same message will reappear on the display until the bed is placed at the right angle, that is, the one requested on the display.

**CARDIAC CHAIR AND AUTO CONTOUR MICRO SWITCH REPLACEMENT****Required Tools:**

Phillips Screwdriver

#1 Phillips screwdriver

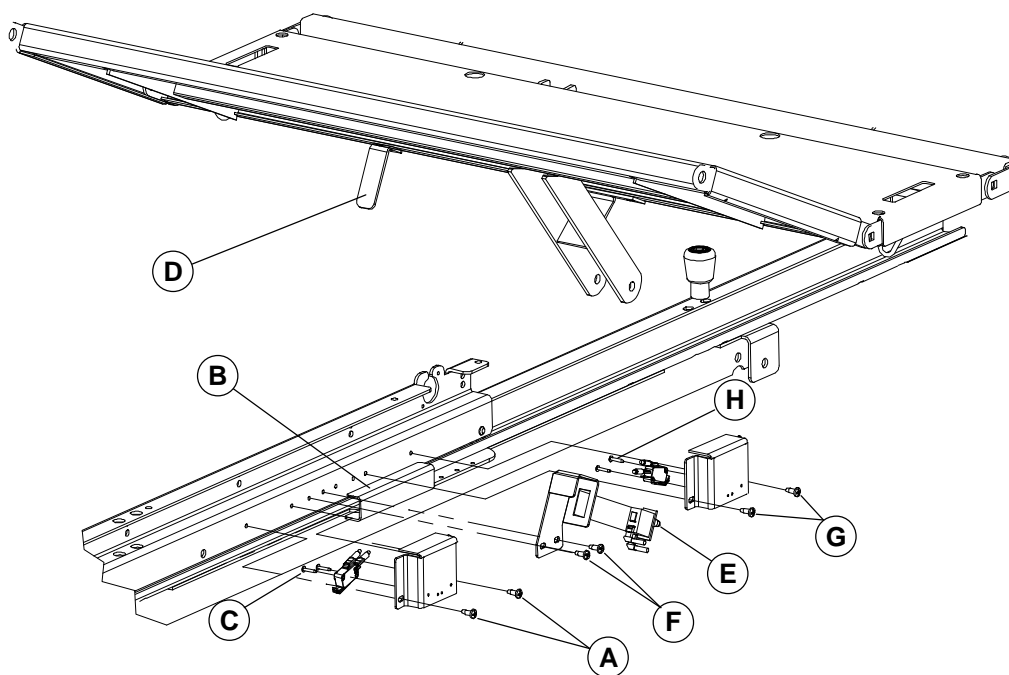**Procedure:**

**Figure 4.3G**

1. Raise the bed fully up and apply the brakes.
2. Raise the thigh section and fold the foot section back toward the head end of the bed.

- **Cardiac Chair Micro Switch**

This switch tells the control board that the Fowler has reached the preset angle for the Cardiac Chair position.

3. Unplug the bed power cord from the wall socket.
4. Using a Phillips screwdriver, remove the two screws (A) holding the support to the frame and remove the support.
5. Remove the cable wires from the switch. Note their location so they will be connected correctly to the new switch.

**NOTE**


The mounting position of the support may be adjusted thanks to the oblong holes on the support. The proper adjustment of the support enables the switch blade to be activated when it meets the activator (B) during the cardiac chair positioning process.

6. Using a #1 Phillips screwdriver, remove the two screws (C) holding the switch to the support and remove the defective switch.
7. Reverse the above steps to install the new micro switch.
8. Test the Cardiac Chair positioning for proper operation before returning the bed to service.  
**Note:** The test of the Cardiac Chair must be done with a weight of at least 50 lb laid on the bed.

- **Micro Switch Common to the Cardiac Chair and Auto Contour Mechanism**

This switch tells the control board that the Knee Gatch has reached the preset angle for the Cardiac Chair and auto Contour positions.

9. Remove the cable wires from the switch. Note their location so they will be connected correctly to the new switch.

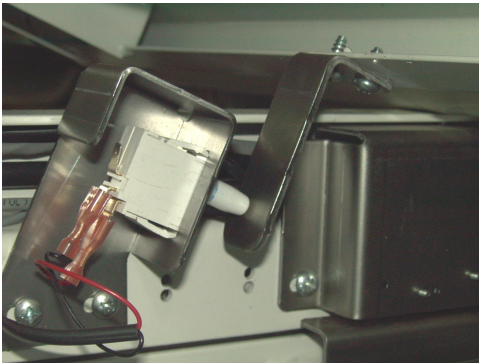

10. Since the support has oblong holes to adjust its position, we will position the thigh section bed in such a way that the correct mounting adjustment will be easy to obtain.
11. Using the Knee Gatch down control, slowly lower the thigh section until only a light contact remains between the activator (D), attached to the knee section, and the switch push button (E). The replacement procedure will start from this point (see opposite illustration).
12. Unplug the bed power cord from the wall outlet.
13. Using a Phillips screwdriver, remove the two screws

(F) holding the support to the frame.

**NOTE**

Mount the support at the same position than the one mentioned at step 10.

14. Press the two switch clips to remove from the support.
15. Reverse the above steps to install the new micro switch.
16. Test the Cardiac Chair and the Auto Contour mechanisms for proper operation before returning the bed to service. **Note:** The Cardiac Chair test must be done with a weight of at least 50 lb laid on the bed.

- **Auto Contour Micro Switch**

When the Fowler is lowered while the bed in the Auto Contour position, this switch tells the control board to start lowering the Knee Gatch.

17. Unplug the bed power cord from the wall outlet.
18. Using a Phillips screwdriver, remove the two screws (G) holding the support to the frame and remove the support.
19. Remove the cable wires from the switch. Note their location so they will be connected correctly to the new switch.
20. Using a #1 Phillips screwdriver, remove the two screws (H) holding the switch to the support.
21. Reverse the above steps to install the new micro switch.
22. Test the Auto Contour positioning for proper operation before returning the bed to service.

## CPR MICRO SWITCH REPLACEMENT

### Required Tools:

#1 Phillips Screwdriver

Phillips Screwdriver

### Procedure:

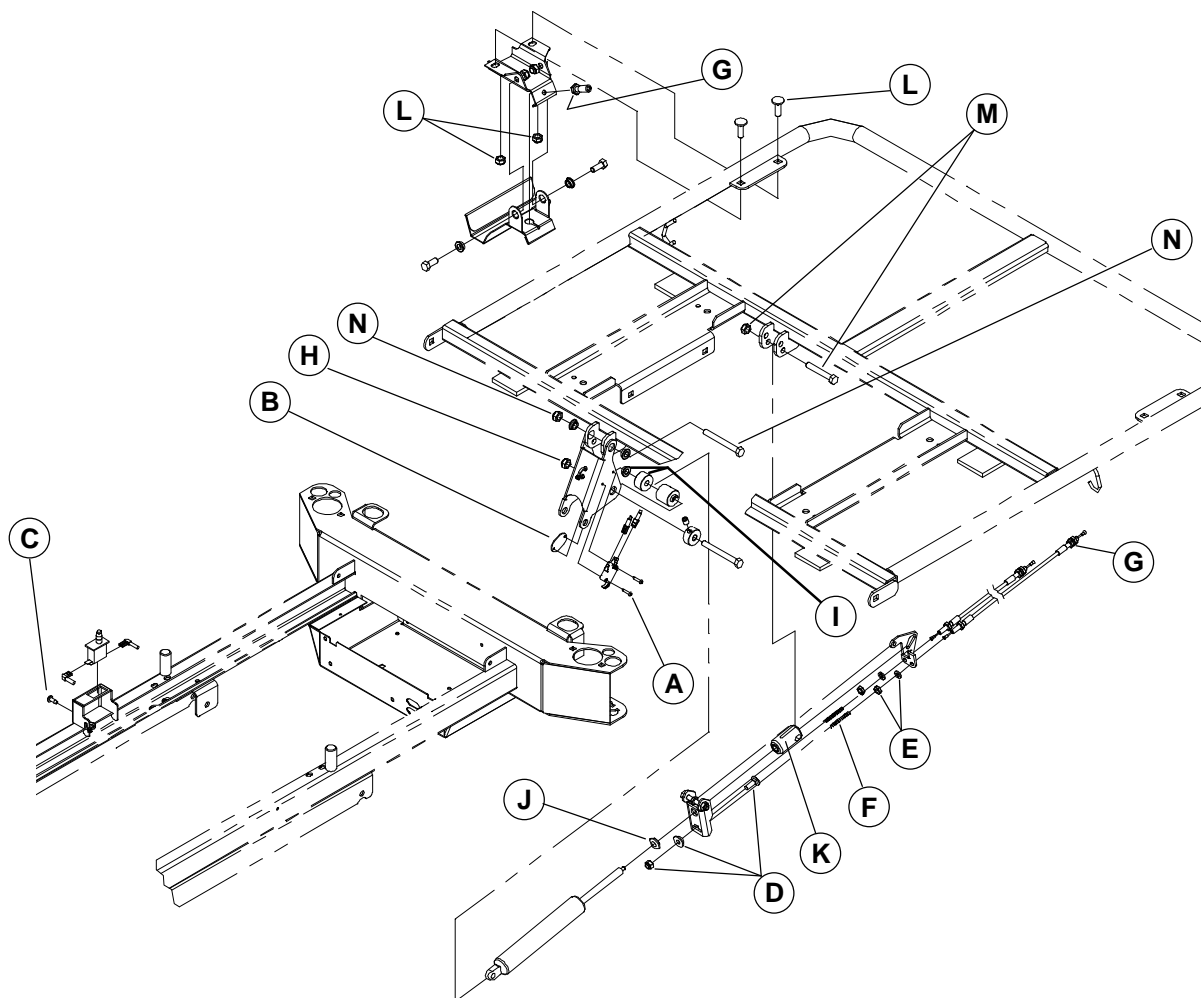


Figure 4.3H

1. Raise the bed fully up and apply the brakes.
2. Raise the head siderails.
3. Raise the head section approximately 45°.
4. Unplug the bed power cord from the wall outlet.

#### • Micro Switch Attached to the Head Section Lever

This micro switch tells the control board that the Fowler is completely lowered so that the resetting of the head section actuator may now start.

5. Using a #1 Phillips screwdriver, remove the two screws (A) holding the micro switch to the head section lever. Recuperate the mounting plate (B). Remove the defective micro switch.
6. Remove the cable wires from the micro switch. Note the location of the wires so they will be connected correctly to the new micro switch.
7. Reverse the above steps to install the new micro switch.
8. Test the CPR positioning before returning the bed to service.

- **Micro Switch Attached to the Frame.**

This micro switch tells the control board that the head section is now reset and the CPR positioning process is completed.

9. Using a Phillips screwdriver, remove the screw (C) holding the support to the frame.
10. Remove the cable wires from the micro switch. Note the location of the wires so they will be connected correctly to the new micro switch.
11. Press the switch clips to remove it from the support.
12. Reverse the above steps to install the new micro switch.
13. Test the CPR positioning before returning the bed to service.

## **CPR ACTIVATION CABLE REPLACEMENT**

### **Required Tools:**

7/16"

### **Procedure:**

#### **NOTE**

Unless otherwise indicated, refer to figure 4.3H on page 45 for the illustration of this procedure reference points.

1. Raise the bed fully up and apply the brakes.
2. Remove the head section plastic molded shell.
3. Raise the head section fully up and remove the head board.
4. Unplug the bed power cord from the wall outlet.
5. Using a 7/16" wrench, remove the nut/sleeve/bolt (D) holding both ends of the activation cable to the fixed lever.
6. Using 7/16" wrench, remove the nut/washer (E) holding the defective cable jacket to the mobile lever. Remove the cable from the lever and recuperate the spring (F). Try as much as possible not to move the other cable from its position.
7. Unscrew completely the two lock nuts (G) holding the other end of the defective cable to the CPR handle. Remove the cable. Make sure there is no free play at the CPR handle once the new cable installed.
8. Reverse the above steps to install the new cable.
9. Test the CPR positioning before returning the bed to service.

## **CPR PNEUMATIC CYLINDER**

### **Required Tools:**

Two 1/2" Wrenches

### **Procedure:**

#### **NOTE**

Unless otherwise indicated, refer to figure 4.3H on page 45 for the illustration of this procedure reference points.

1. Raise the bed fully up and apply the brakes.
2. Remove the head section plastic molded shell.
3. Unplug the bed power cord from the wall outlet.
4. Using two 1/2" wrenches, remove the nut (H) from the bolt holding the lower end of the pneumatic cylinder.
5. Partially remove the bolt until the lower end of the cylinder is free. Recuperate the shoulder spacer/sleeve (I). Disengage the lower end of the cylinder from the remaining sleeve.
6. Using an 11/16" wrench, loosen the lock nut (J) and unscrew the cylinder completely from the end fitting (K). Note the location of the lock nut before loosening it so the new cylinder will be properly screwed into the end fitting.
7. Reverse the above steps to install the new cylinder.
8. Test the CPR mechanism for proper operation before returning the bed to service.

## 4.4 BASE MAINTENANCE PROCEDURES

### BRAKE/STEER PEDAL REPLACEMENT

#### Required Tools

1/2" Socket Wrench

Soft Hammer

#### Procedure:

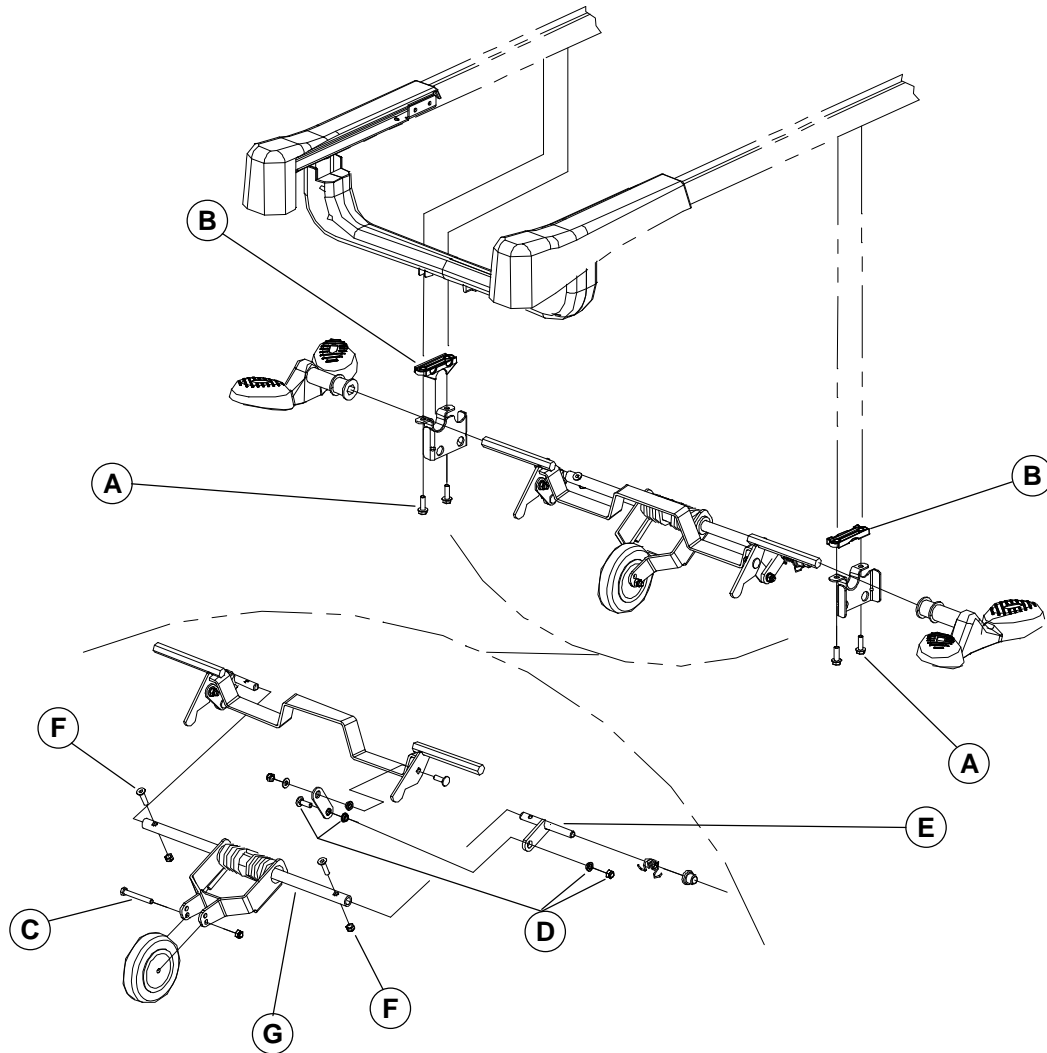


Figure 4.3I

1. Raise the bed fully up.
2. Unplug the bed power cord from the wall outlet.
3. Remove the optional 5th wheel hood.
4. Position the brake/steer pedal to neutral.
5. Using a 1/2" socket wrench, remove the two bolts (A) holding the 5th wheel support on the side of the defective pedal and lay the support down. Recuperate the molded spacer (B).
6. Using a soft hammer, remove the pedal from the activation lever shaft.
7. Reverse the above steps to install the new pedal.

## **5th WHEEL CASTER REPLACEMENT**

### **Required Tools**

Two 1/2" Wrenches

### **Procedure:**

#### **NOTE**

Unless otherwise indicated, refer to figure 4.3I on page 47 for the illustration of this procedure reference points.

1. Raise the bed fully up.
2. Unplug the bed power cord from the wall outlet.
3. Remove the optional 5th wheel hood.
4. Position the brake/steer pedal to neutral.
5. Using two 1/2" wrenches, remove the nut/bolt (C) holding the caster to the swing arm.
6. Reverse the above steps to install the new pedal.

## **5th WHEEL ASSEMBLY REPLACEMENT**

### **Required Tools:**

1/2" Socket Wrench

Soft Hammer

### **Procedure:**

#### **NOTE**

Unless otherwise indicated, refer to figure 4.3I on page 47 for the illustration of this procedure reference points.

1. Raise the bed fully up.
2. Unplug the bed power cord from the wall outlet.
3. Remove the optional 5th wheel hood.
4. Position the brake/steer pedal to neutral.
5. Using a 1/2" socket wrench, remove the two nuts/shoulder spacers/washers/bolts (C, fig. 4.3J, page 49) holding the brake rods to the locking levers on both sides of the mechanism. If the optional four-wheel brake system is present on the bed, also remove the fasteners holding the two other brake rods to the locking levers.
6. Using a 1/2" socket wrench, remove the two bolts (A) holding each side of the mechanism to the base. Recuperate the molded spacers (B). Remove the assembly.
7. Using a soft hammer remove the two brake/steer pedals from the activation lever shafts.
8. Remove the two locking levers (D, fig. 4.3J, page 49).
9. Reverse the above steps to install the new 5th wheel assembly.
10. Test the 5th wheel assembly before returning the bed to service.



**5th WHEEL SWING ARM ASSEMBLY REPLACEMENT****Required Tools**

1/2" Socket Wrench

Two 1/2" wrenches

3/16" Allen Key

**Procedure:****NOTE**

Unless otherwise indicated, refer to figure 4.3I on page 47 for the illustration of this procedure reference points.

1. Raise the bed fully up.
2. Unplug the bed power cord from the wall outlet.
3. Remove the optional 5th wheel hood.
4. Position the brake/steer pedal to neutral.
5. Using a 1/2" socket wrench, remove the two bolts (A) holding the left or right 5th wheel support to the base. Lay the support down and recuperate the molded spacer (B).
6. Using two 1/2" wrenches, remove the nut/shoulder spacers/bolt (D) holding the torsion lever (E) to the lower part of the counter lever.
7. Using a 1/2" wrench and a 3/16" Allen key, remove the two nuts/Allen screws (F) holding the torsion levers (E) to the torque shaft (G).

**NOTE**

At reassembly, screw down the Allen screws before installing the nuts.


8. Disengage the swing arm assembly from the torsion levers and remove it.
9. Using two 1/2" wrenches, remove the nut/bolt (C) holding the caster to the swing arm and replace the caster on the new swing arm assembly.
10. Reverse the above steps to install the new swing arm assembly.
11. Test the 5th wheel before returning the bed to service.

**BED CASTER REPLACEMENT****Required Tools:**


Jack Stand

Long Nose Pliers

1/2" Wrench

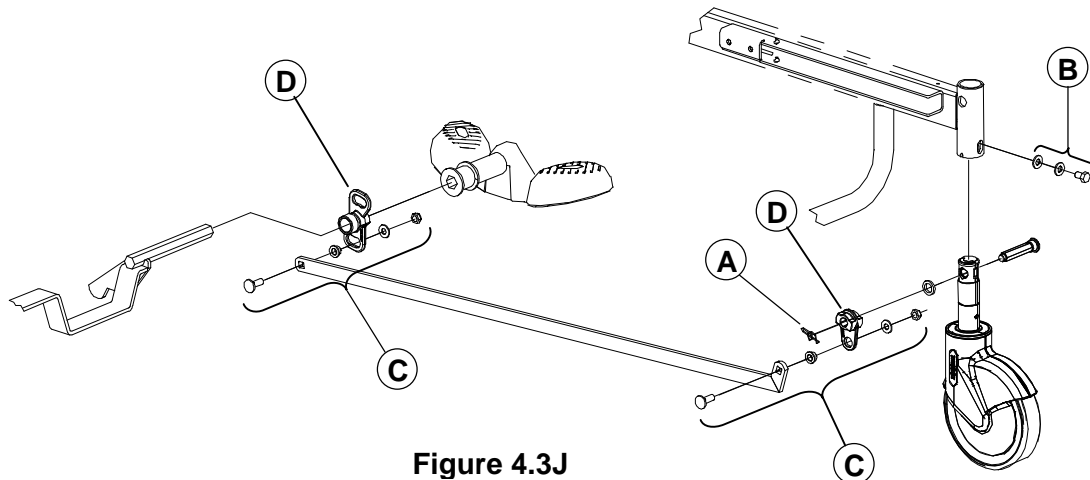**Procedure:****Figure 4.3J**

1. Raise the bed fully up.
2. Position the brake/steer to neutral
3. Install a jack stand under the frame where the defective caster is.
4. Remove the wheel cover.
5. Lower the bed until the caster is six inches off the ground.
6. Unplug the bed power cord from the wall outlet.

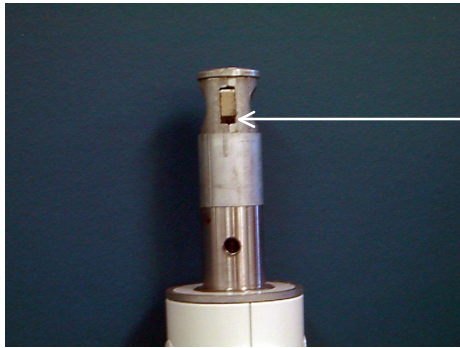

7. If the caster is part of the braking mechanism, remove, using long-nose pliers, the rue ring cotter/washer/locking axle (A) linking the caster shaft to the locking lever and brake rod.

**NOTE**

The shaft of a caster that is part of the braking mechanism must be properly oriented before mounting it to the base. The opposite illustration shows the side of the shaft that should be facing the inside of the bed. It is the side showing the largest hole below the cam in the neutral position.

8. Using a 1/2" wrench, remove the bolt/washers (B) holding the caster to the base.

**NOTE**

Before tightening the bolt (B), make sure the shaft is completely inserted into the mounting socket.

9. Reverse the above steps to install the new caster.

## **BRAKE ROD REPLACEMENT**

### **Required Tools:**

1/2" Wrench

### **Procedure:**


**NOTE**

Unless otherwise indicated, refer to illustration 4.3J on page 49 for the illustration of this procedure reference points

1. Raise the bed fully up.
2. Using a 1/2" wrench, remove the nut/shoulder spacer/washer/bolt (C) holding each end of the brake rod.
3. Remove the defective rod.
4. Reverse the above steps to install the new brake rod.
5. Test the brakes before returning the bed to service.

APPENDIX A: CIRCUIT DIAGRAMS


A) Can Board: Motor Control - Page 1 of 2

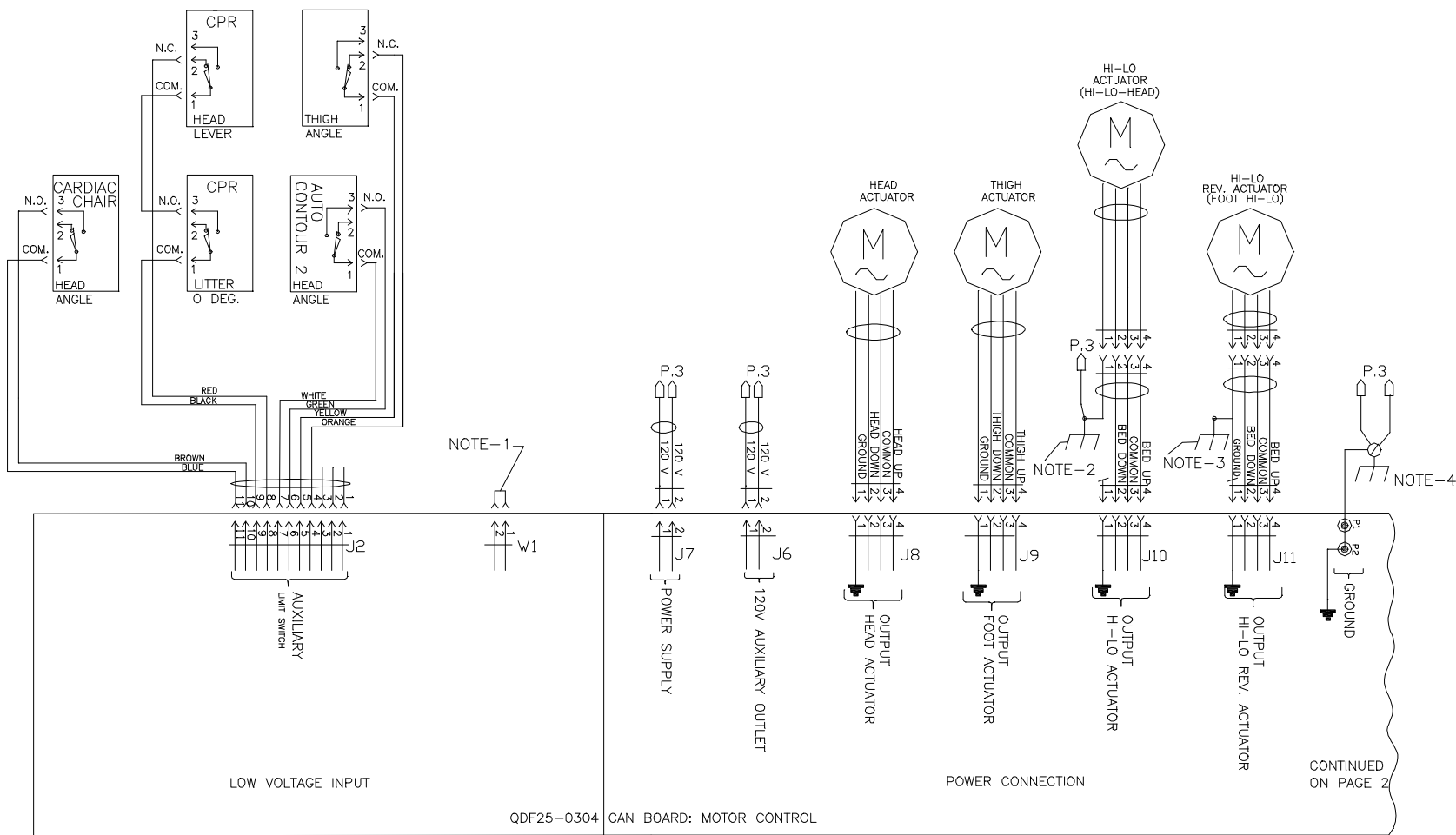

CAN BOARD: MOTOR CONTROL  
QDF25-0304

SW2 SWITCH POSITIONS						
BED MODEL	SW2					
	1	2	3	4	5	6
FL25	OFF	OFF	OFF	OFF	ON	OFF

B) Can Board: Motor Control - Page 2 of 2

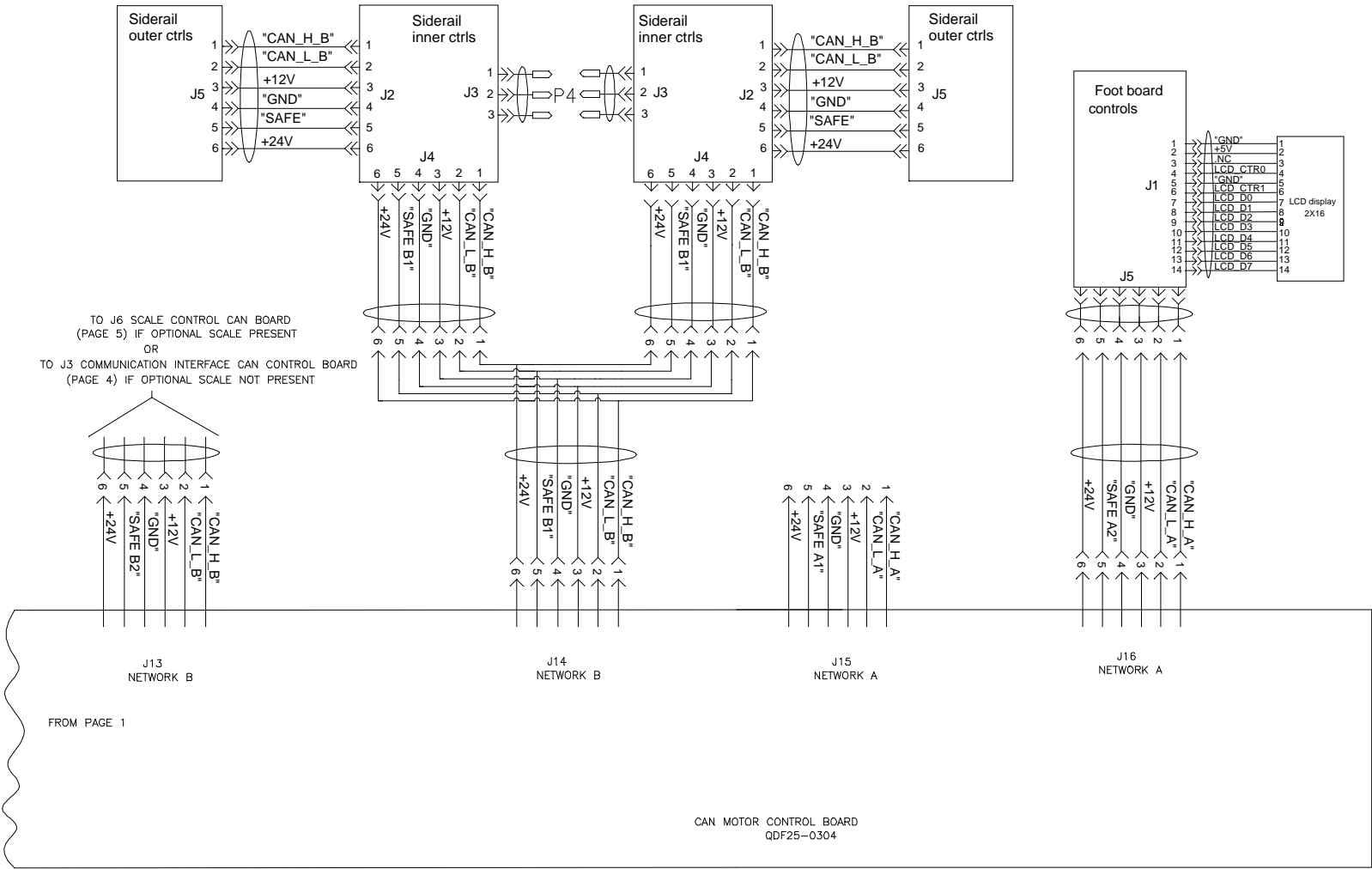

CAN BOARD-HEAD SIDERAIL OUTER CTRL PANEL  
QDF21-1151

POSITION DES INTERRUPTEURS SW6					
EMPLACEMENT DE LA CARTE	SW6				
	1	2	3	4	5
GAUCHE	OFF	ON	ON	ON	ON
DROIT	ON	ON	ON	OFF	OFF

CAN BOARD-HEAD SIDERAIL INNER CTRL PANEL  
QDF21-1169


SW13 SWITCH POSITIONS			
BOARD LOCATION	SW13		
	1	2	3
LEFT	OFF	OFF	OFF
RIGHT	ON	OFF	OFF

CAN BOARD-SIDERAIL CTRL PANEL/FOOT END CTRL PANEL  
QDF21-1154


SW23 SWITCH POSITIONS					
BOARD LOCATION	SW23				
	1	2	3	4	5
FOOT END	ON	OFF	ON	ON	ON

**B) Power Supply**

# 120V Power Supply

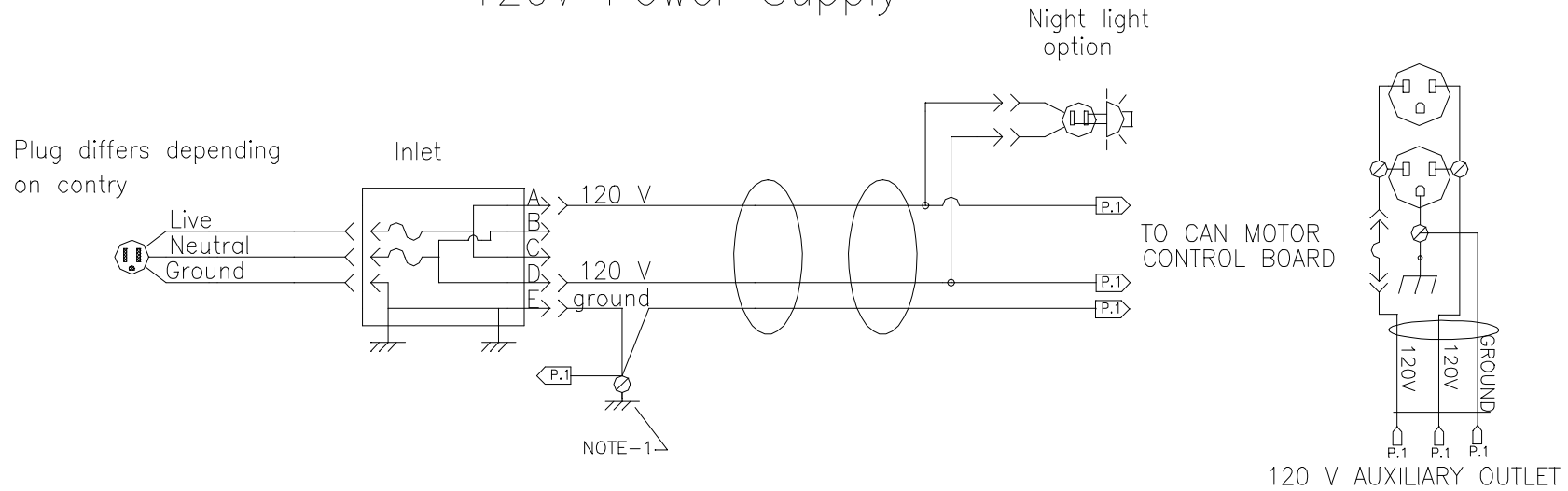


# 100V-200V-220V-240V Power Supply

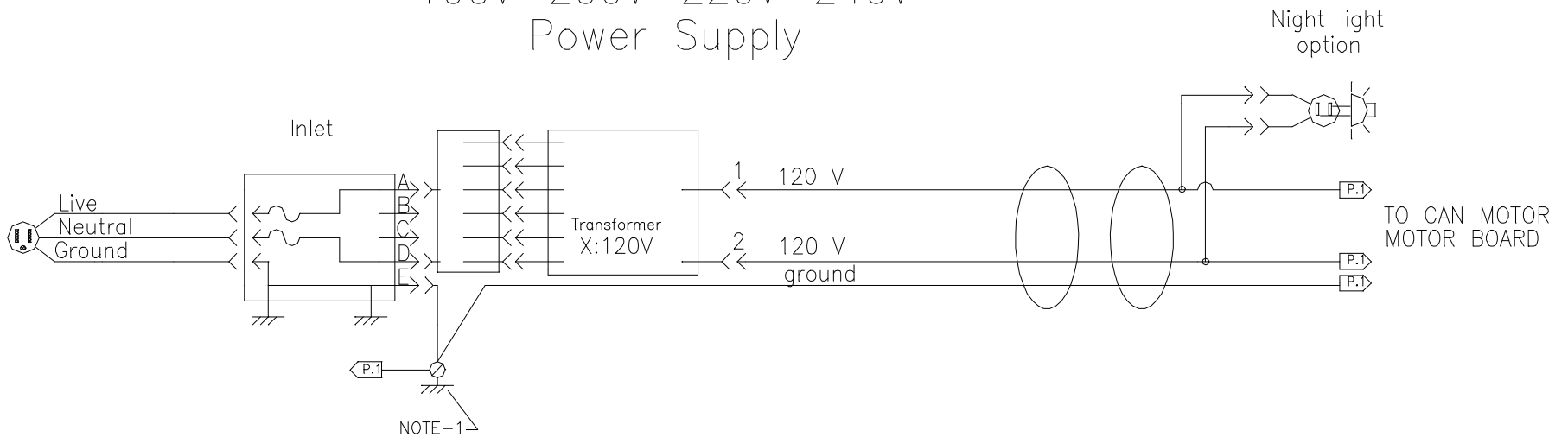




D) Optional Communication Interface

OPTIONAL COMMUNICATION INTERFACE

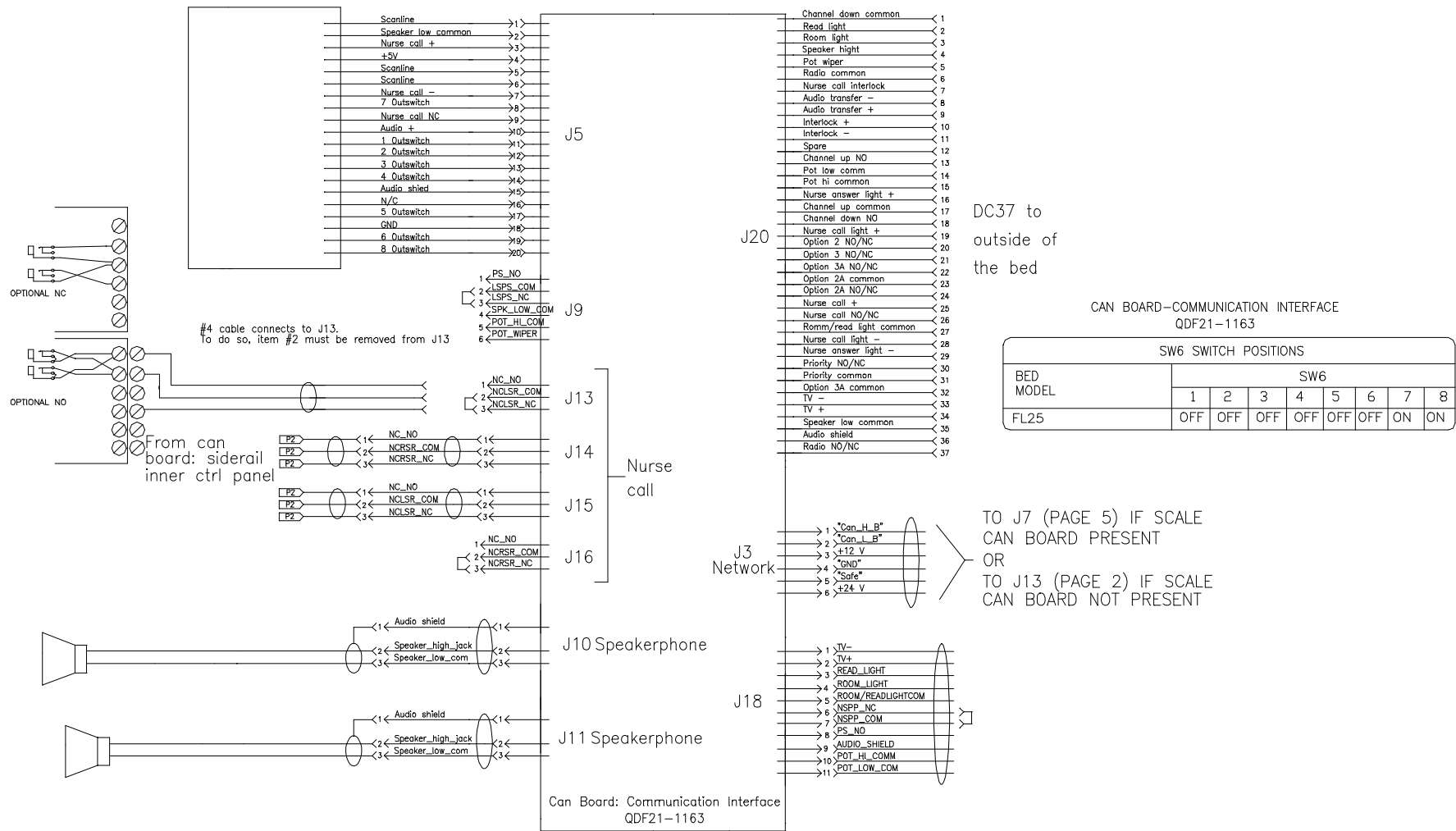

E) Optional Scale

OPTIONAL SCALE

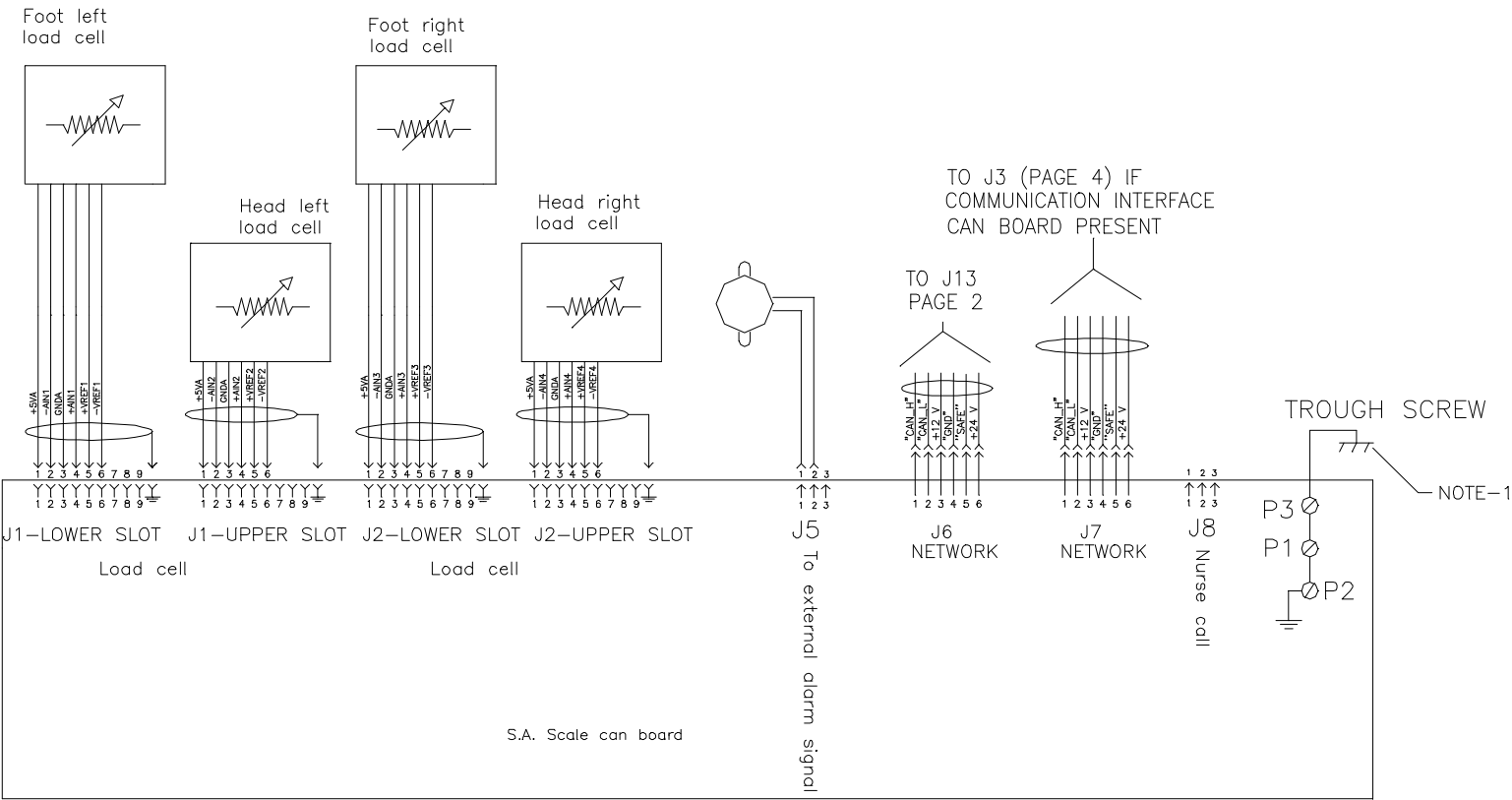

IF BOARD QDF21-1163  
IS PRESENT

S.A. SCALE CAN BOARD  
25-0492

BED MODEL	SW1 SWITCH POSITIONS					
	SW1					
FL25	1	2	3	4	5	6
	OFF	OFF	OFF	OFF	OFF	OFF

IF BOARD QDF21-1163  
IS NOT PRESENT

S.A. SCALE CAN BOARD  
25-0492

BED MODEL	SW1 SWITCH POSITIONS					
	SW1					
FL25	1	2	3	4	5	6
	ON	OFF	OFF	OFF	ON	ON