

stryker®


Parts Manual

Medical


Transport Stretcher
Model ST104



0753-006-110

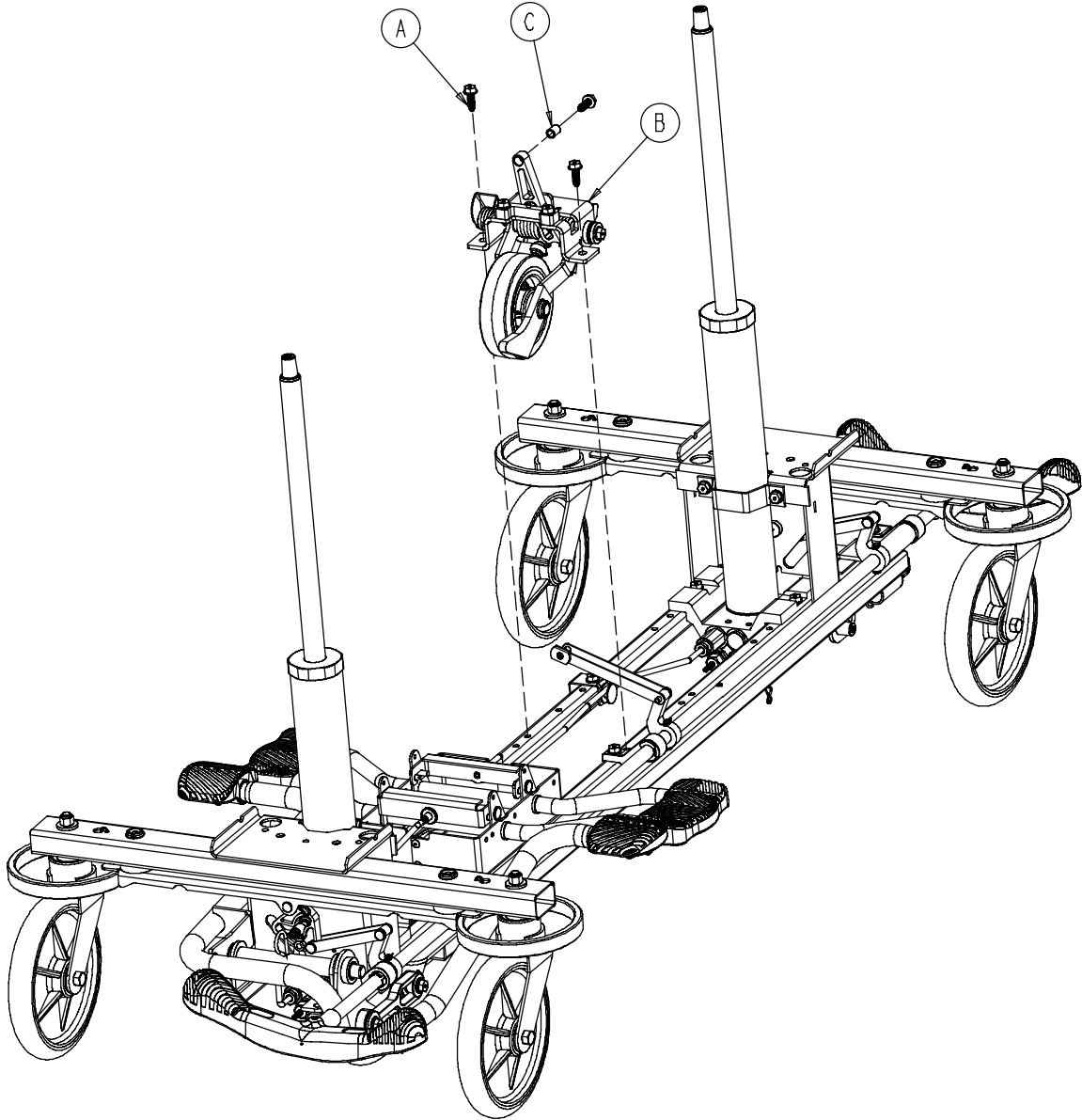


A	0023-288-000	3
B	0753-006-130	1
C	0753-006-148	1

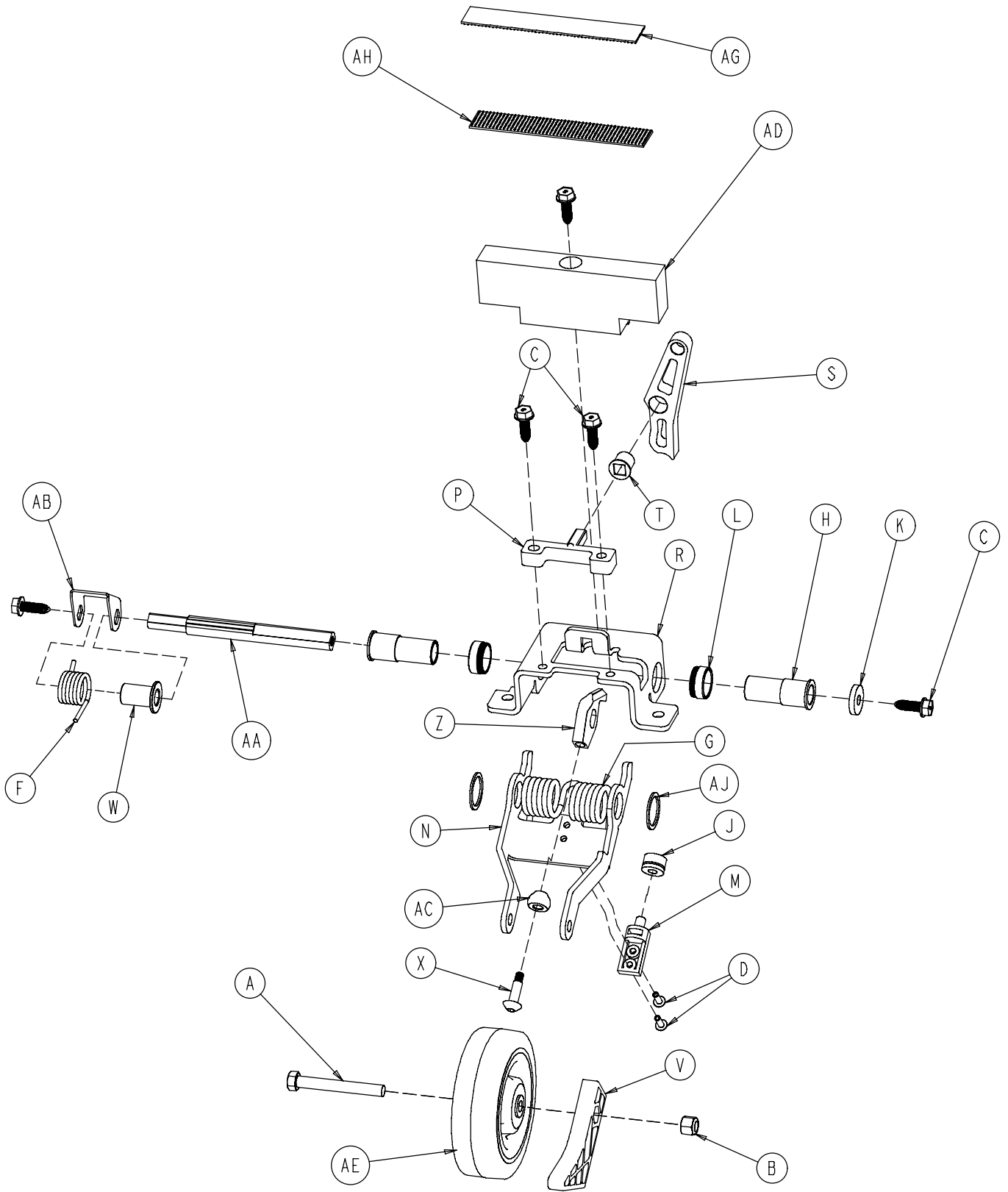


0753-006-130

A	0003-083-000	1
B	0016-035-000	1
C	0023-288-000	5
D	0025-050-000	2
E	0038-525-000	1
F	0753-006-074	1
G	0753-006-075	1
H	0753-006-097	2
J	0753-006-106	1
K	0753-006-108	1
L	0753-006-115	2
M	0753-006-120	1
N	0753-006-126	1
P	0753-006-133	1
R	0753-006-134	1
S	0753-006-142	1
T	0753-006-143	1
V	0753-006-149	1
W	0753-006-152	1
X	0753-006-153	1
Z	0753-006-198	1
AA	0753-006-223	1
AB	0753-006-227	1
AC	0753-006-277	1
AD	0753-010-045	1
AE	1210-001-147	1
AG	0753-006-280	1
AH	0753-006-281	1
AJ	0011-360-000	2

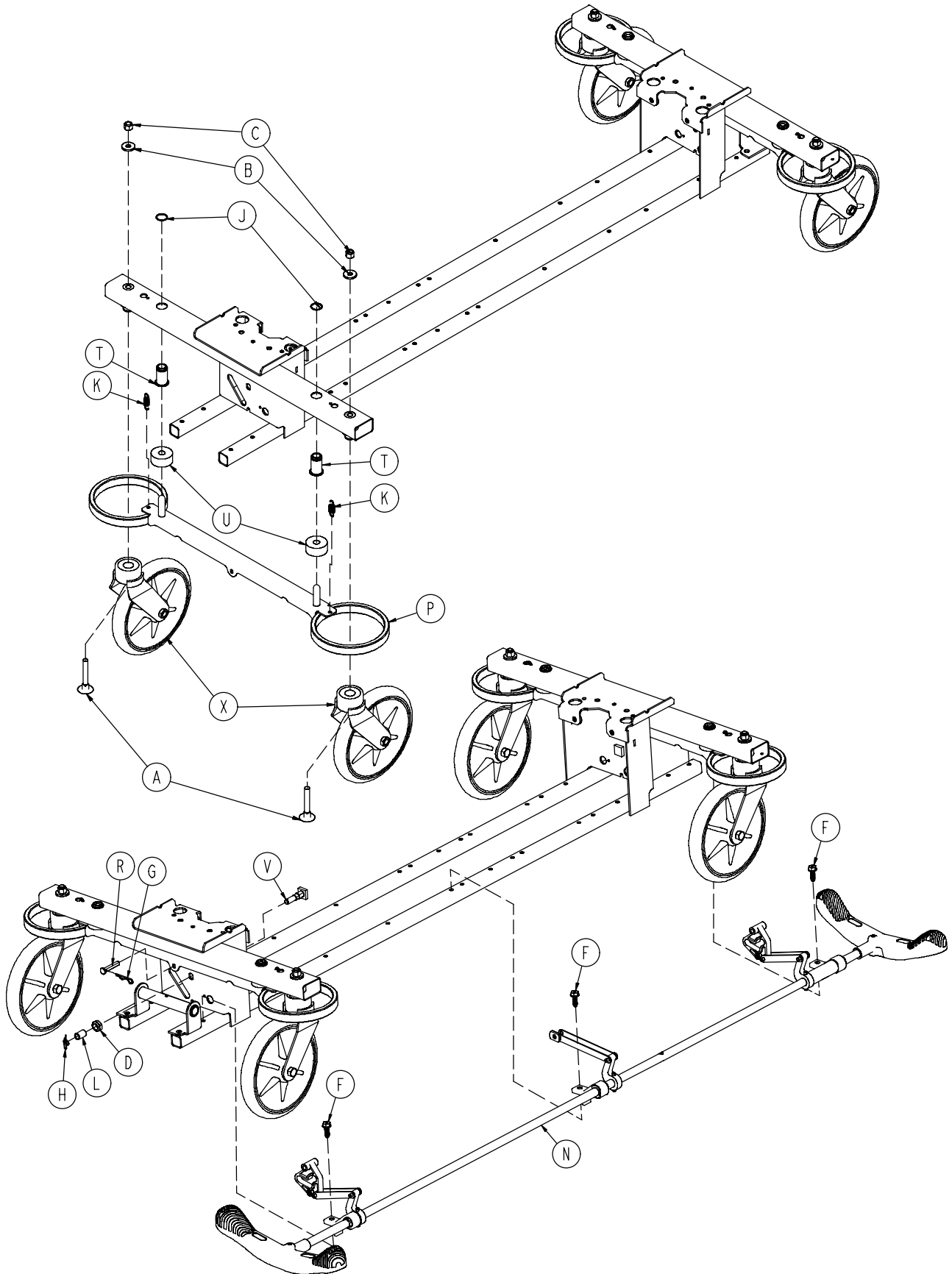


0753-003-105

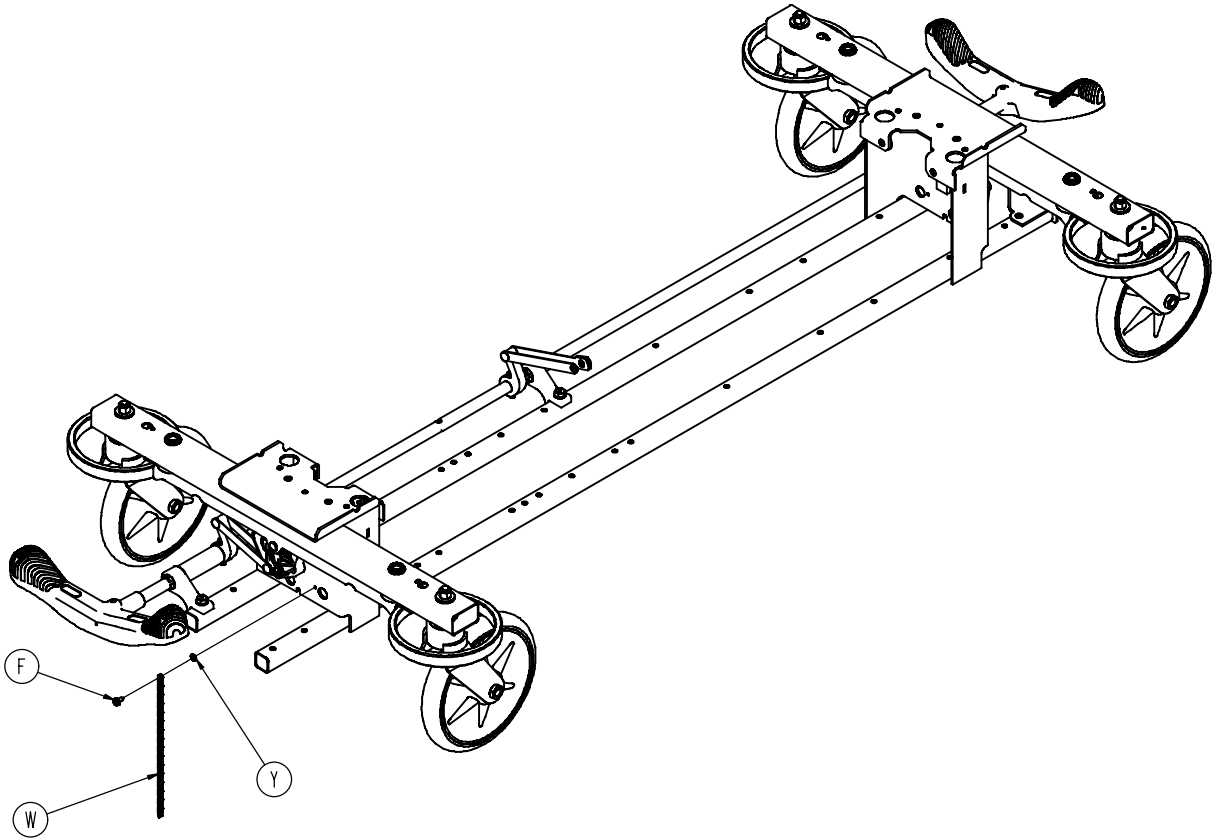


A	0005-044-000	4
B	0011-262-000	4
C	0016-035-000	4
D	0016-049-000	2
E	0023-288-000	3
F	0003-364-000	1
G	0027-012-000	2
H	0027-022-000	2
J	0028-037-000	4
K	0038-439-000	4
L	0081-037-000	2
M	0753-001-001	1
N	0753-003-001	1
P	0753-003-006	2
R	0753-003-066	2
T	0753-003-079	4
U	0753-003-121	4
V	0753-003-230	2
W	3001-200-052	1
X	0753-010-020	4
Y	0013-018-000	1

0753-003-001

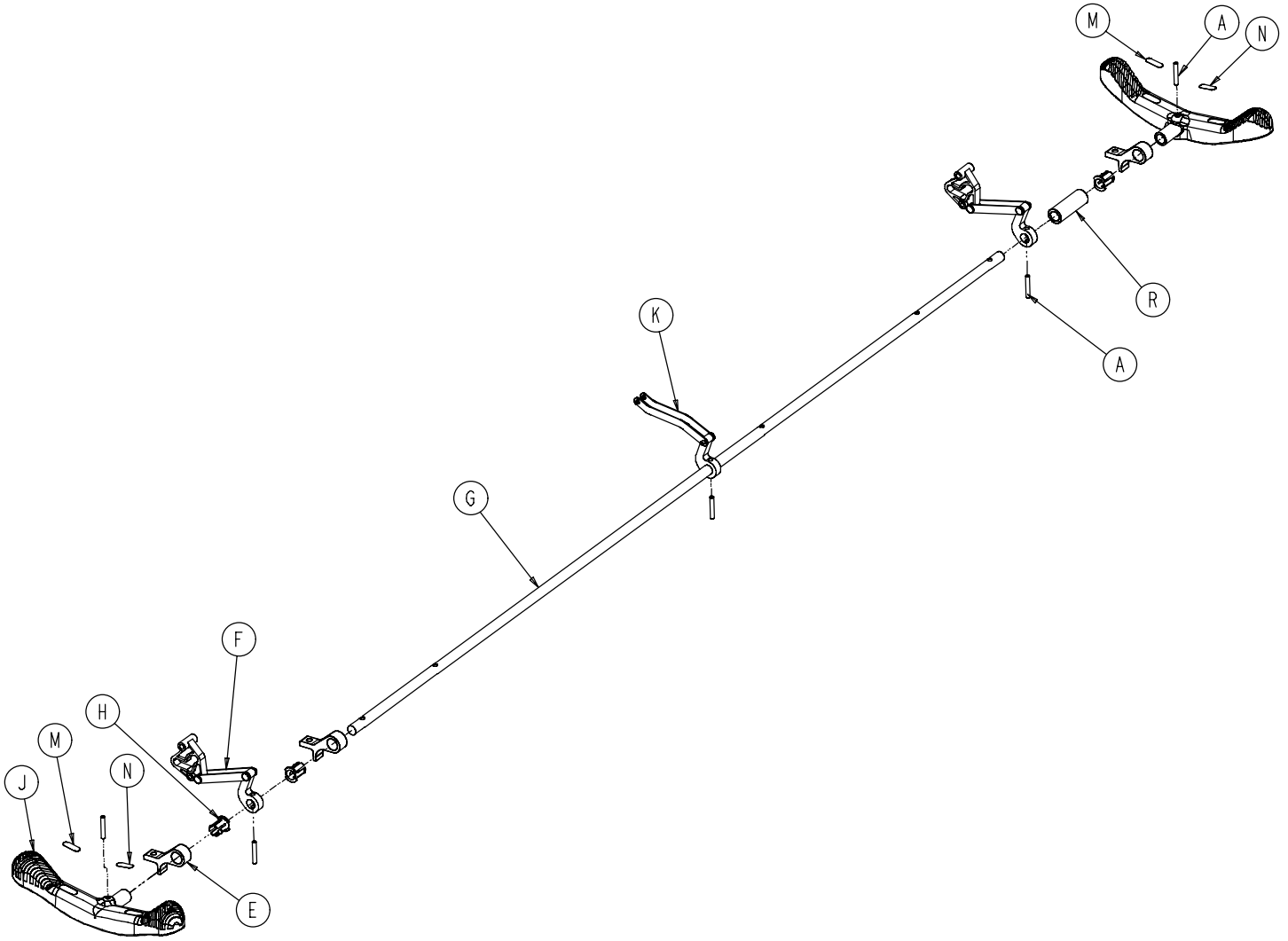


A	0026-067-000	2
E	0753-003-004	3
F	0753-003-010	2
G	0753-003-014	1
H	0753-003-015	3
J	0753-003-099	2
K	0753-006-135	1
M	1210-201-335	2
N	1210-201-336	2
R	0753-003-133	1

0753-003-010

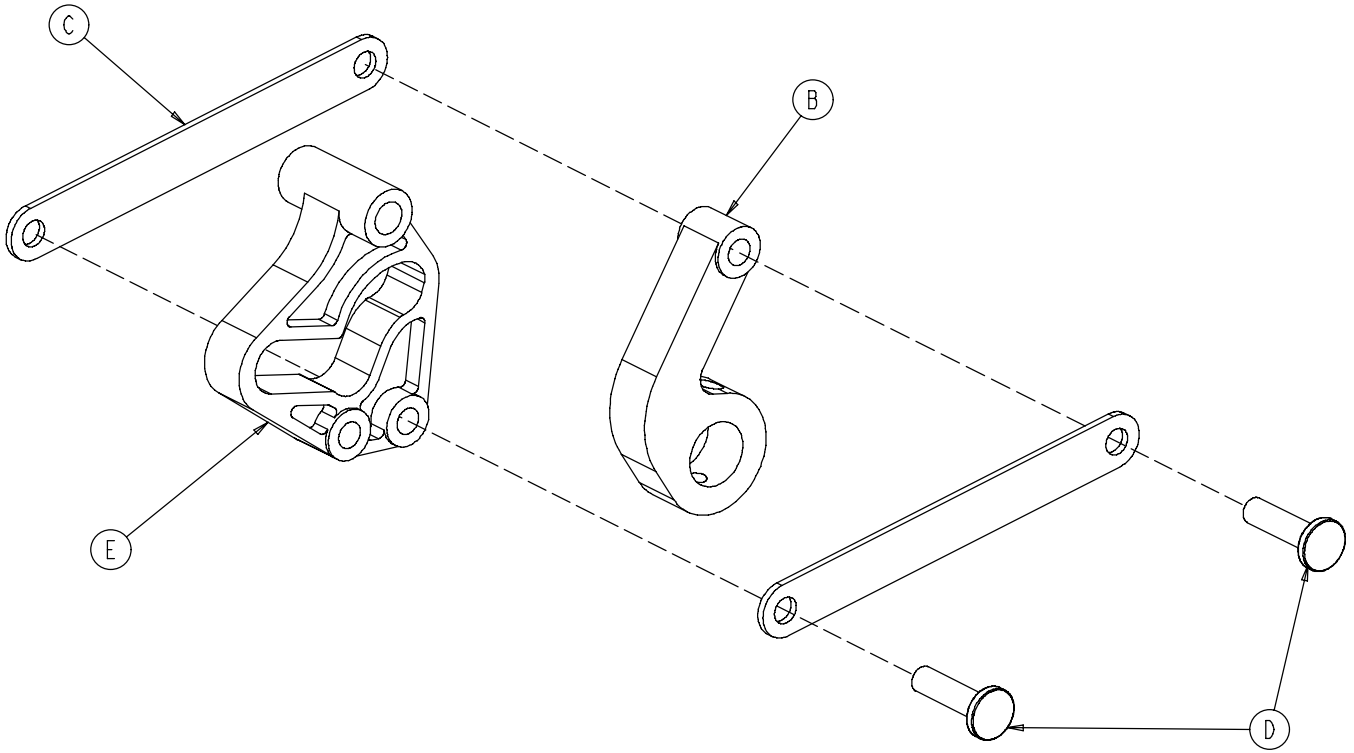


B	0753-003-011	1
C	0753-003-061	2
D	0753-003-098	2
E	0753-003-102	1

0753-006-135

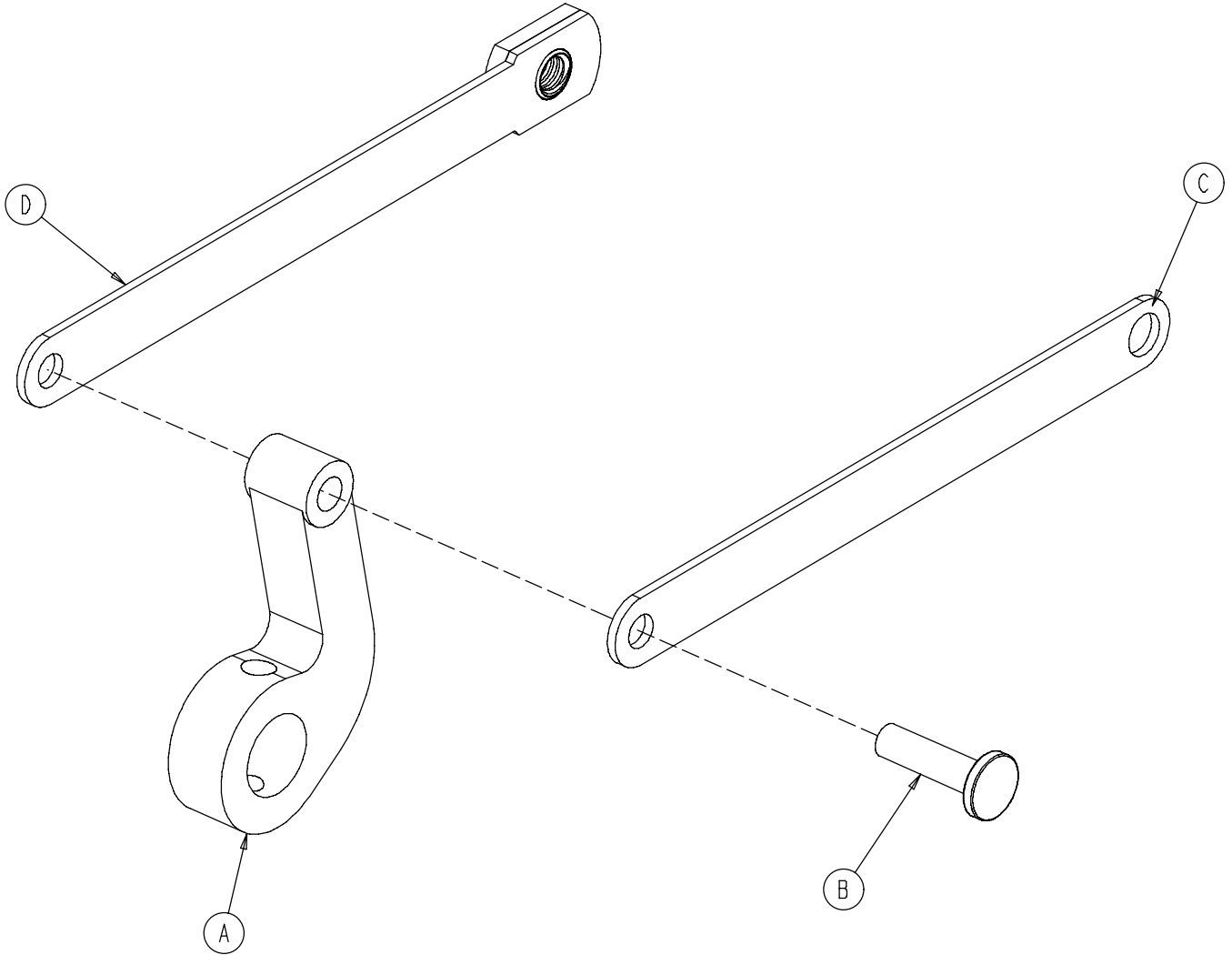


A	0753-003-011	1
B	0753-003-098	1
C	0753-006-122	1
D	0753-006-147	1


0753-010-220

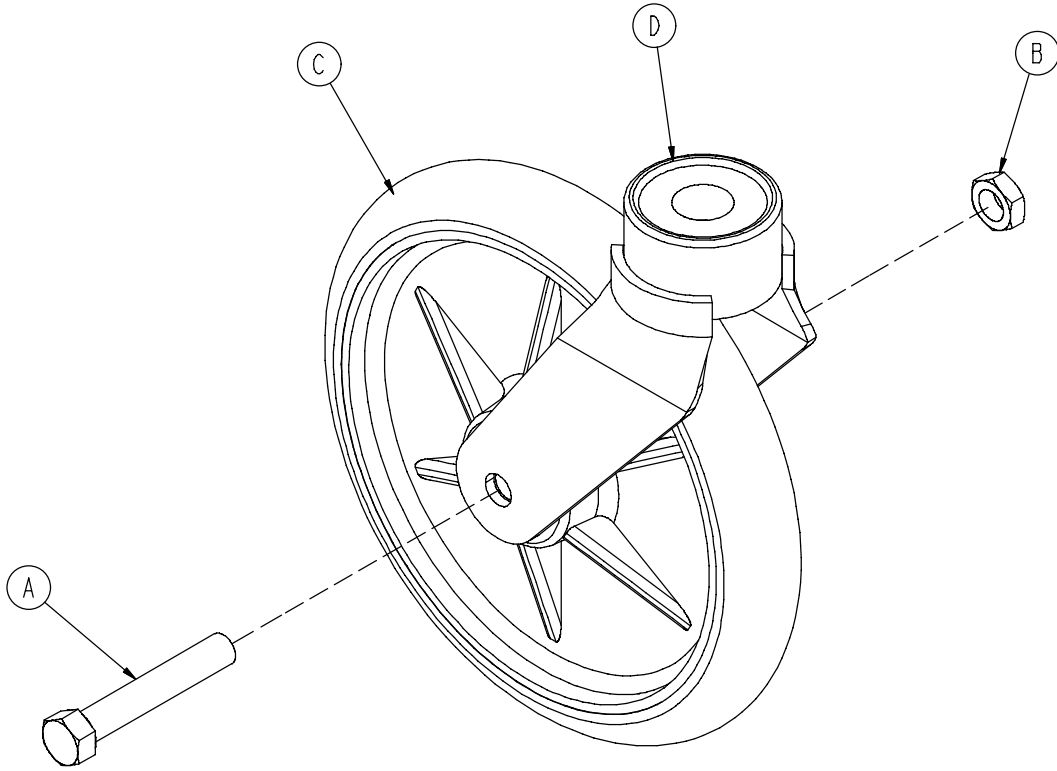

A	0003-099-000	1
B	0016-060-000	1
C	0753-003-215	1
D	0753-010-021	1

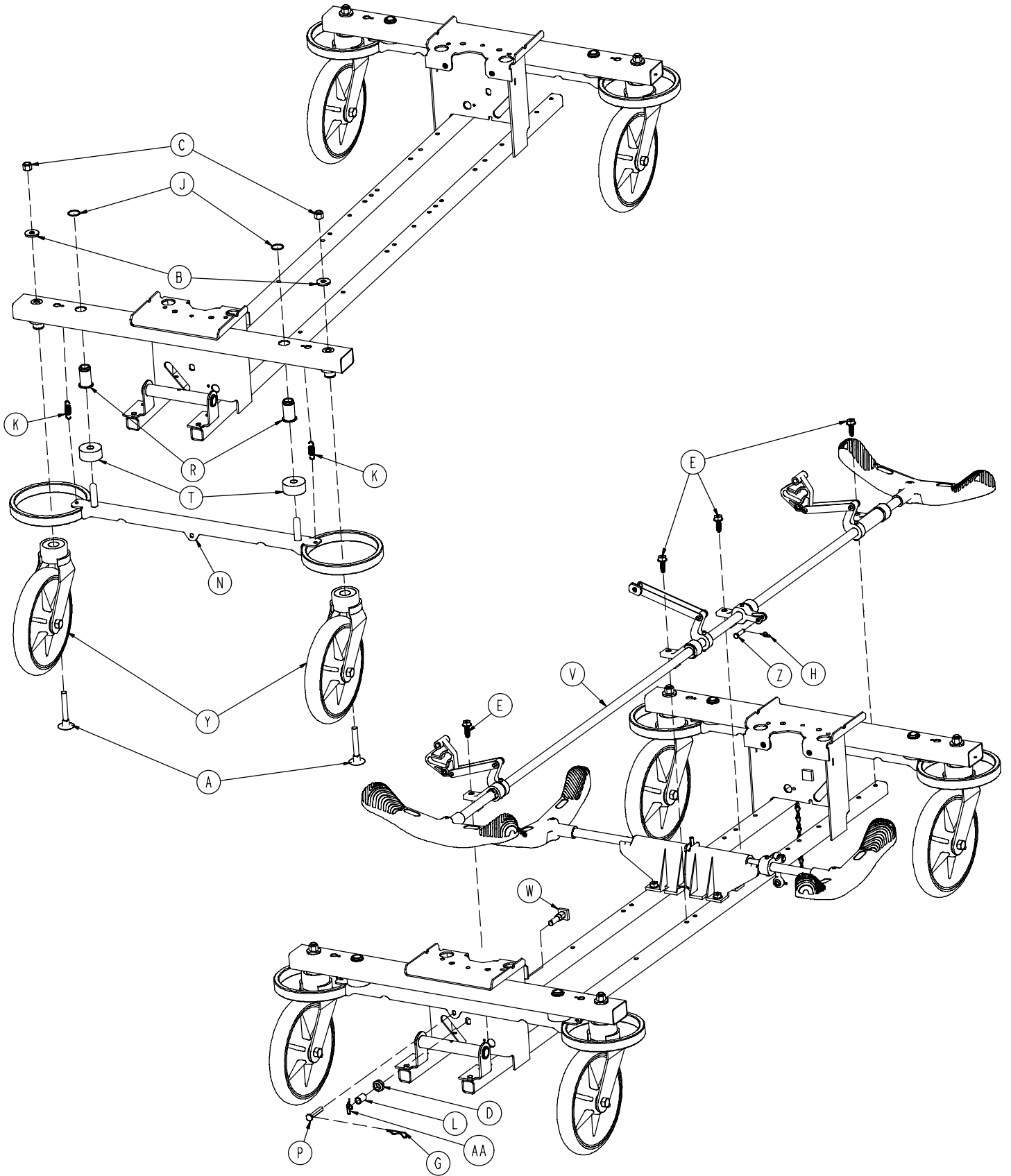


0753-003-120

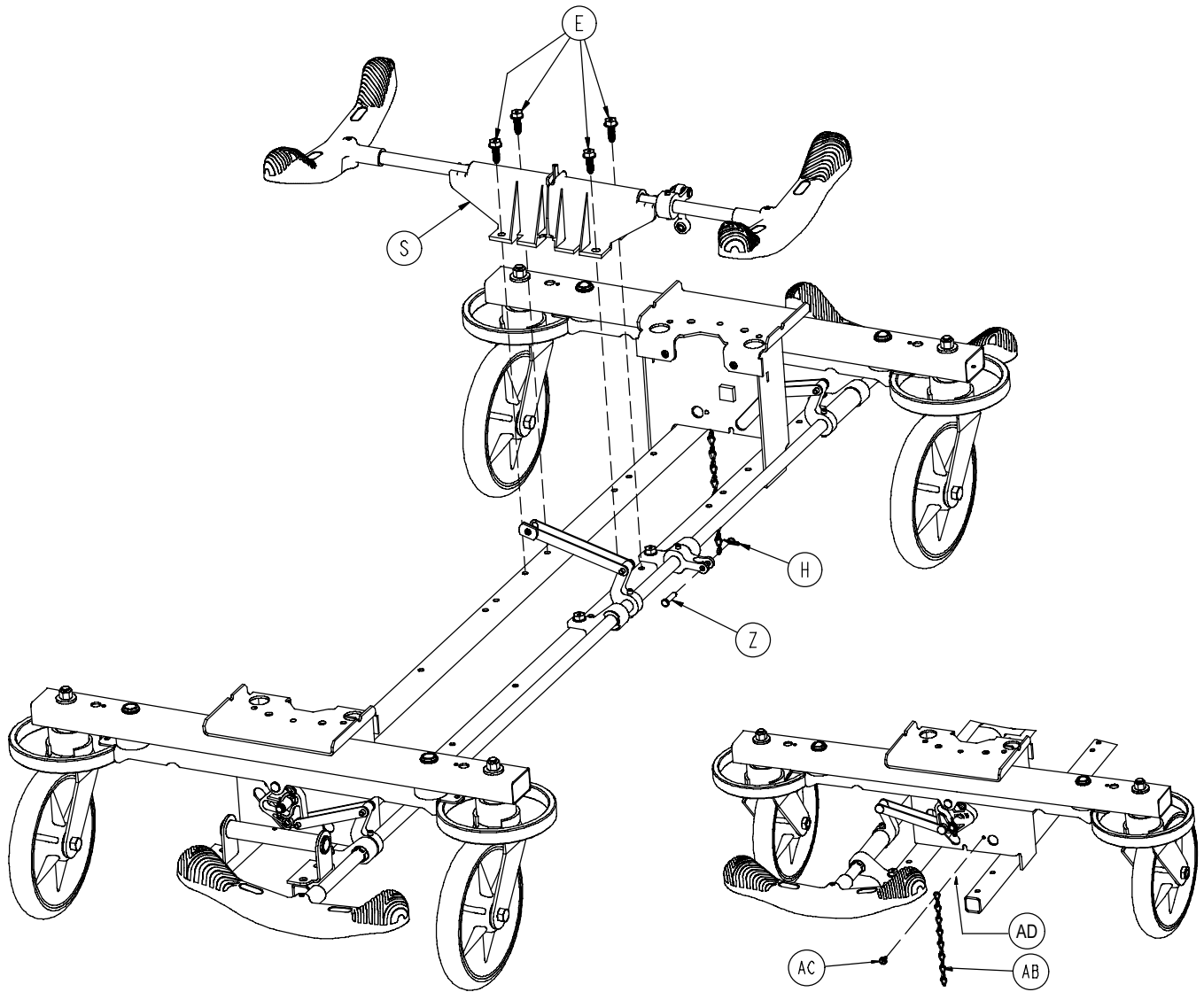


A	0005-044-000	4	P	0753-003-066	2
B	0011-262-000	4	R	0753-003-079	4
C	0016-035-000	4	S	0753-003-210	1
D	0016-049-000	2	T	0753-003-121	4
E	0023-288-000	8	V	0753-003-125	1
G	0027-012-000	2	W	0753-003-330	2
H	0027-020-000	1	Y	0753-010-020	4
J	0028-037-000	4	Z	0026-340-000	1
K	0038-439-000	4	AA	0027-022-000	2
L	0081-037-000	2	AB	3001-200-052	1
M	0753-001-001	1	AC	0003-364-000	1
N	0753-003-006	2	AD	0013-018-000	1

0753-003-125

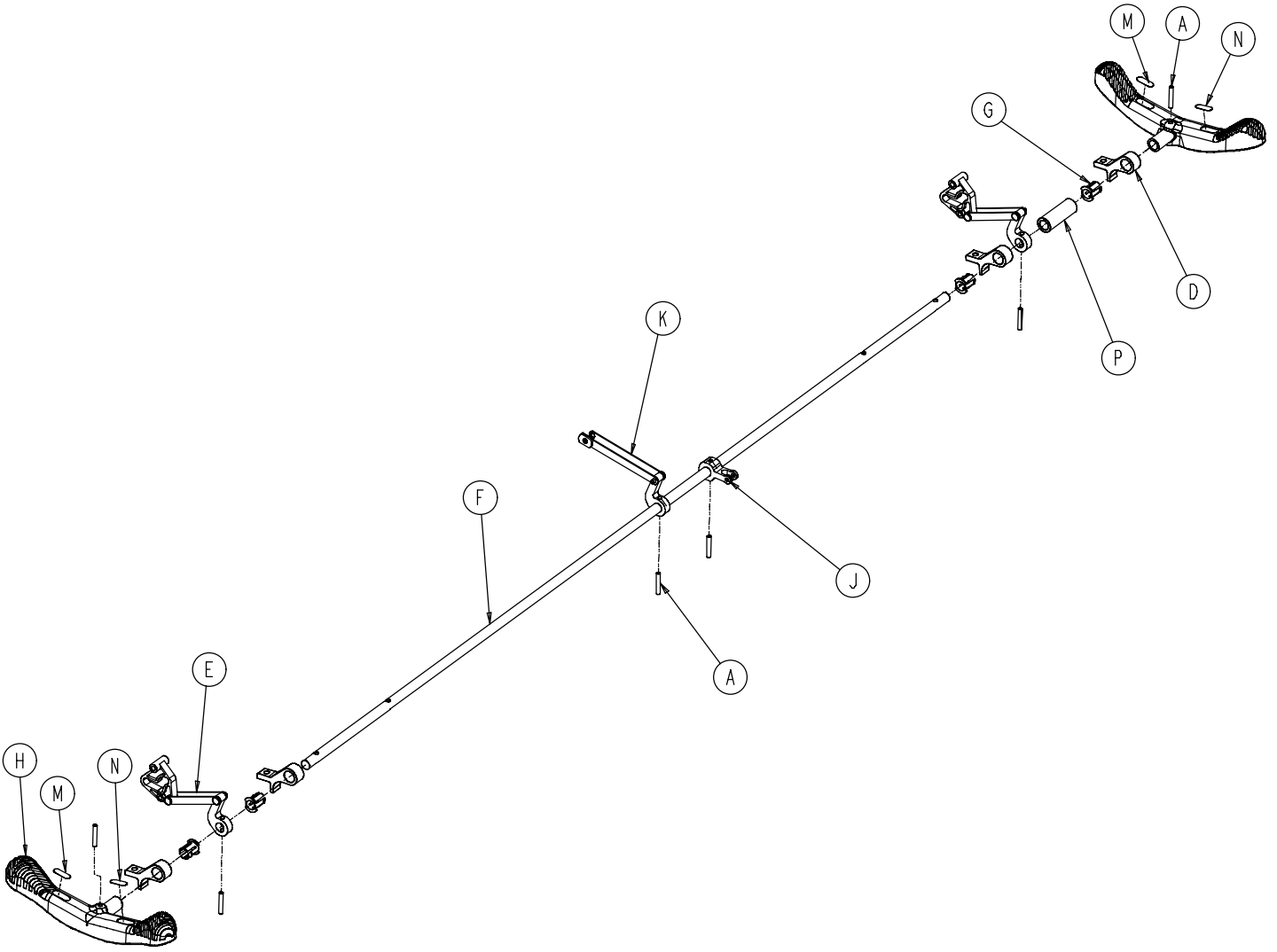



A	0026-067-000	6
D	0753-003-004	4
E	0753-003-010	2
F	0753-003-014	1
G	0753-003-015	4
H	0753-003-099	2
J	0753-003-112	1
K	0753-006-135	1
M	1210-201-335	2
N	1210-201-336	2
P	0753-003-133	1

0753-003-210



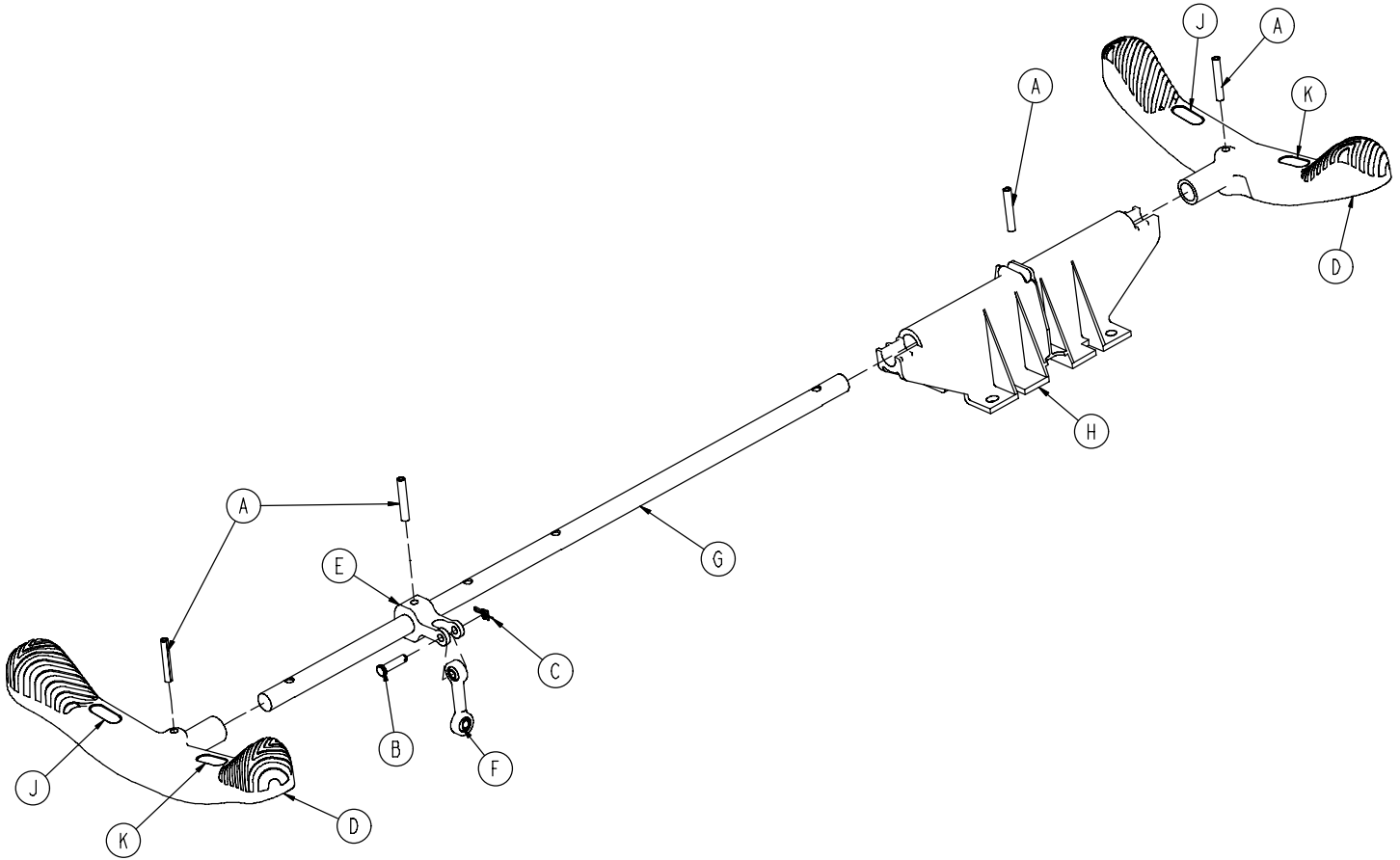


A	0026-067-000	4
B	0026-340-000	1
C	0027-020-000	1
D	0753-003-099	2
E	0753-003-112	1
F	0753-003-117	1
G	0753-003-119	1
H	0753-003-214	1
J	1210-201-335	2
K	1210-201-336	2

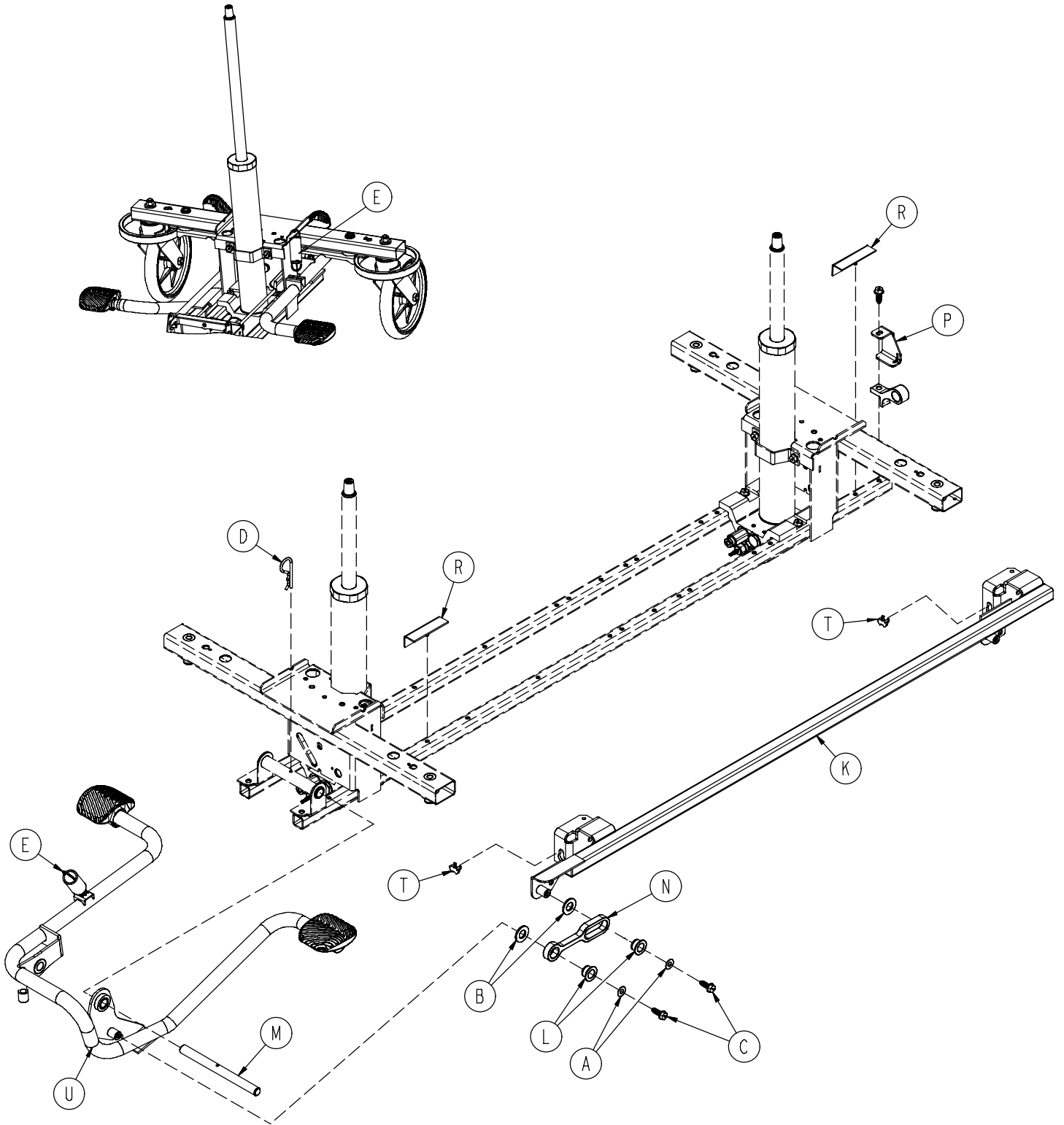

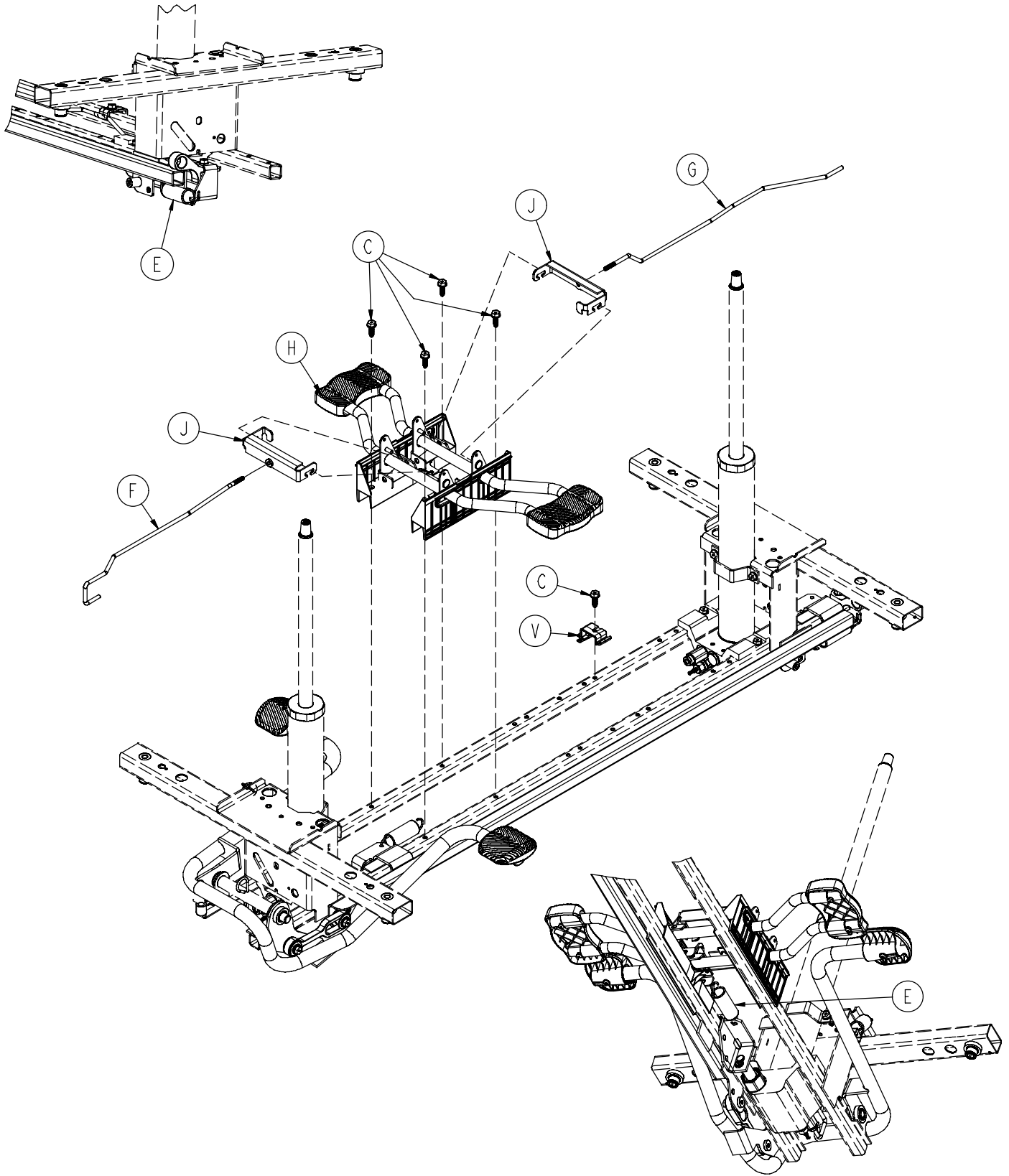


0753-005-200

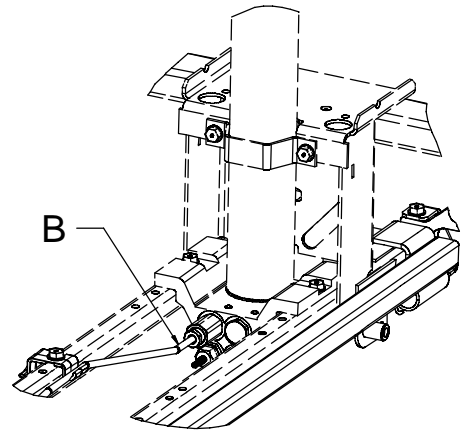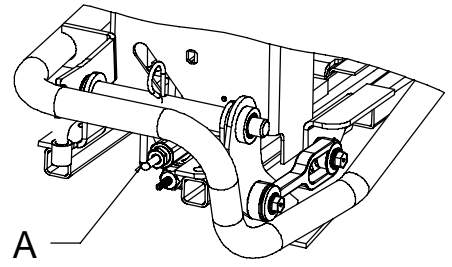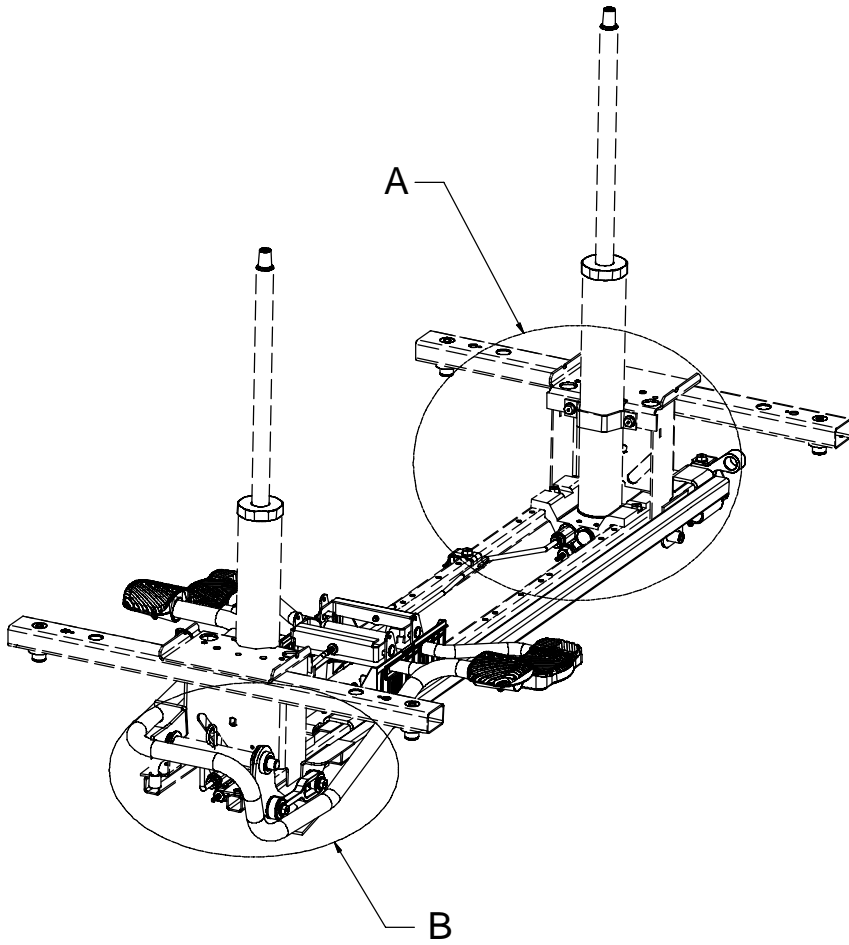


A	0011-023-000	2
B	0014-071-000	2
C	0023-288-000	7
D	0027-031-000	1
E	0038-497-000	3
F	0753-004-014	1
G	0753-004-015	1
H	0753-004-101	1
J	0753-004-132	2
K	0753-005-037	1
L	0753-005-044	2
M	0753-005-074	1
N	0753-005-075	1
P	0753-005-087	1
R	0753-005-088	2
T	0753-005-089	2
U	0753-005-185	1
V	0753-010-215	1

0753-004-101

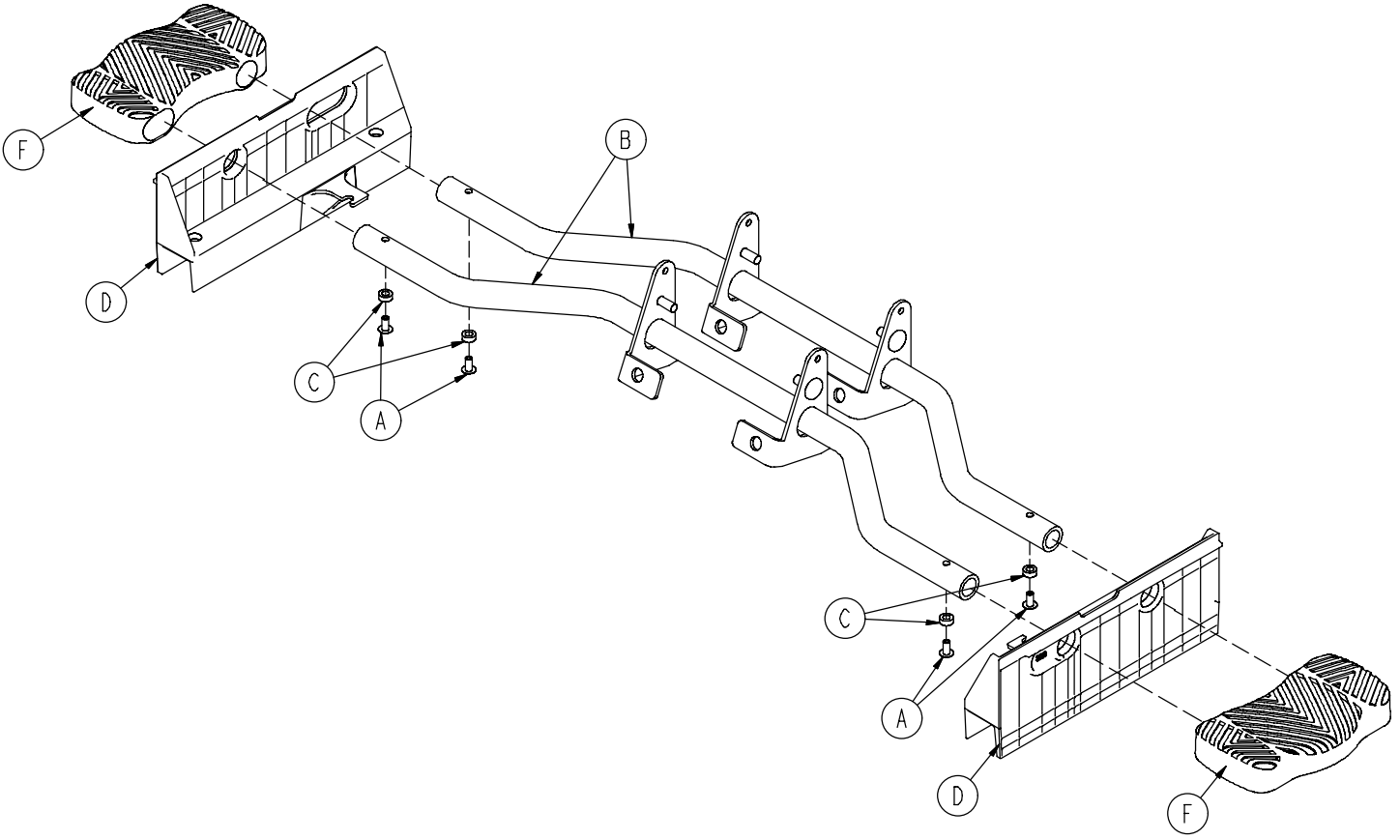


A	0025-079-000	4
B	0753-004-004	2
C	0753-004-029	4
D	0753-004-320	2
F	1061-201-127	2

0753-005-185

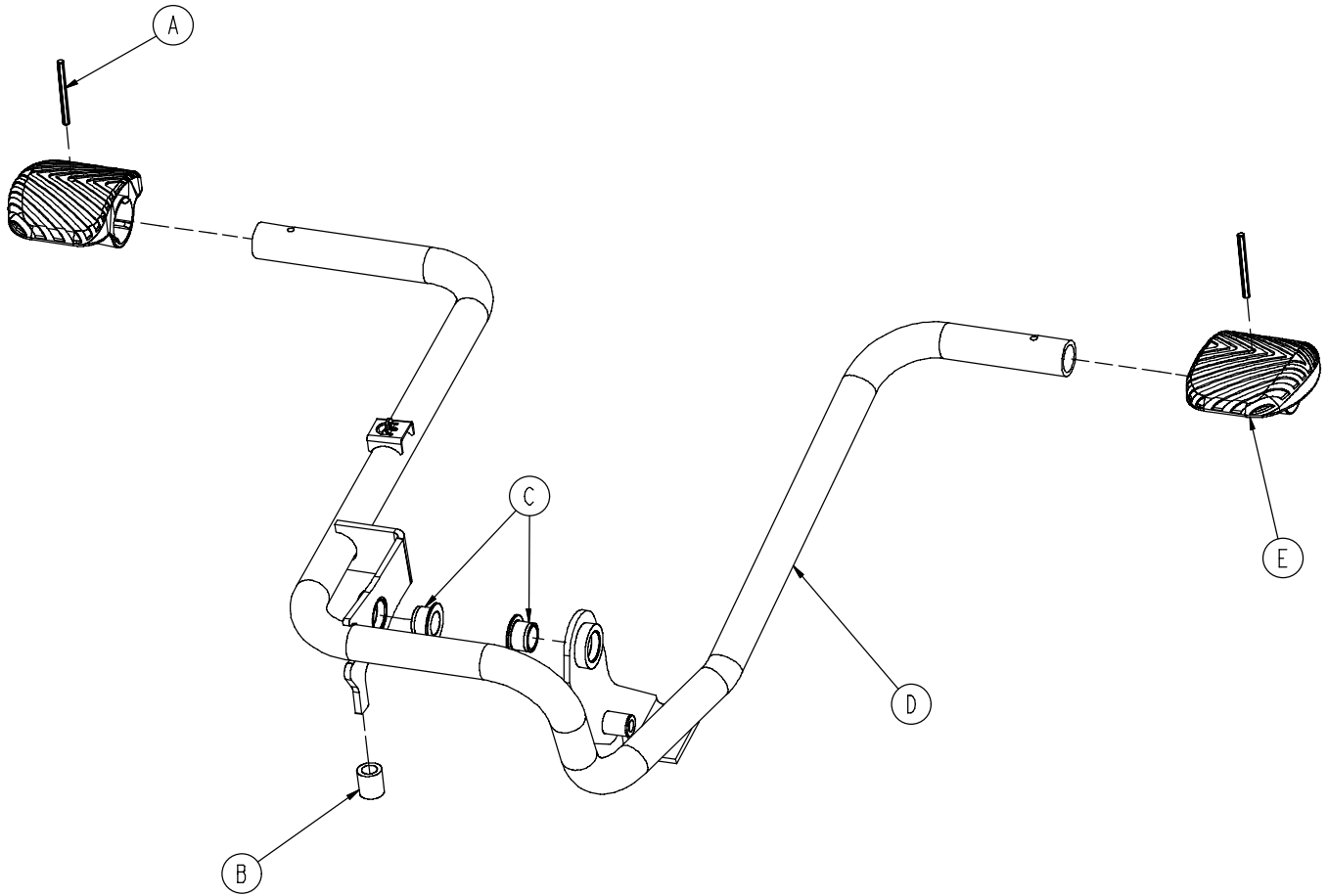


A	0026-343-000	2
B	0715-001-140	1
C	0753-005-044	2
D	0753-005-180	1
E	0753-201-126	2

0753-002-180





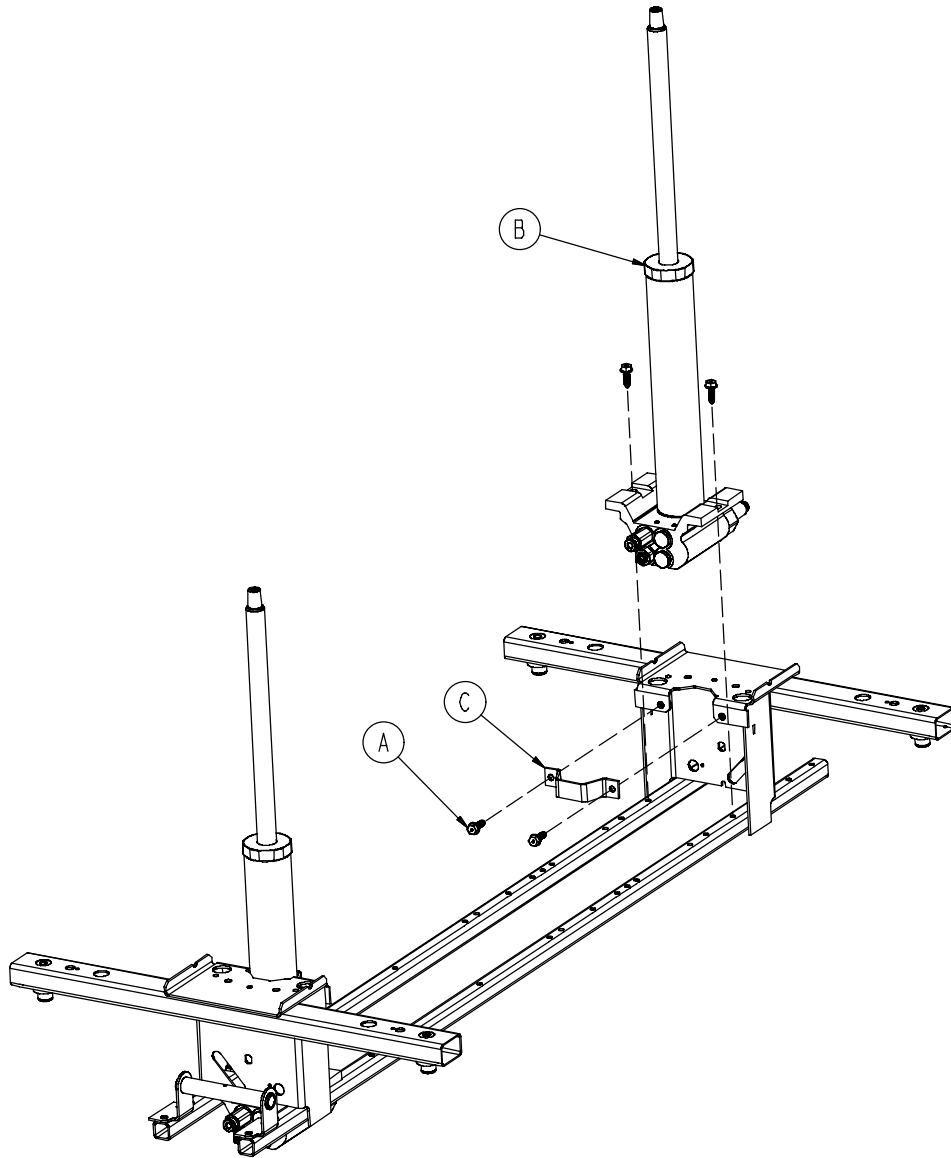


A	0023-288-000	8
B	0753-002-170	2
C	0753-010-007	2

0753-002-075





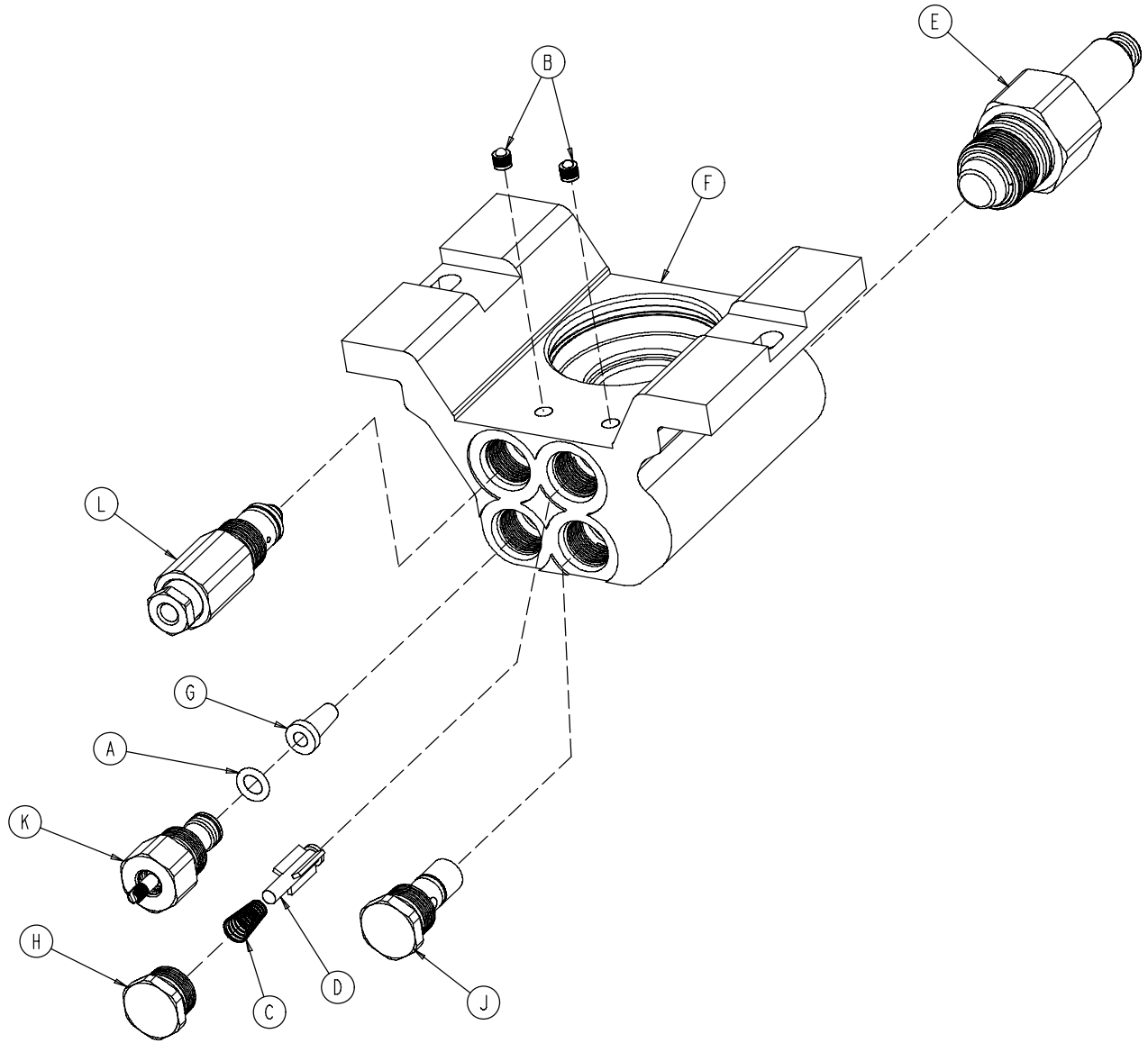


A	0046-004-000	1
B	0048-021-000	2
C	0390-002-134	1
D	0715-001-341	1
E	0753-002-003	1
F	0753-002-010	1
G	0753-002-019	1
H	0753-002-051	1
J	0753-002-052	1
K	0753-002-063	1
L	0753-002-065	1

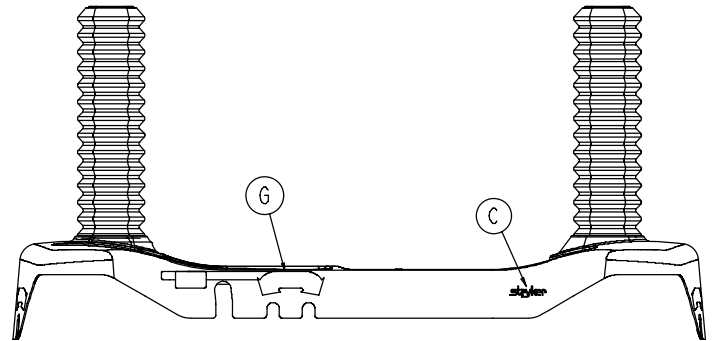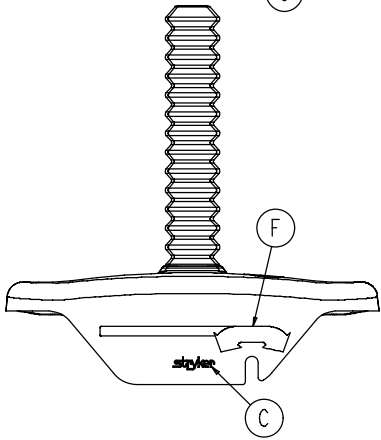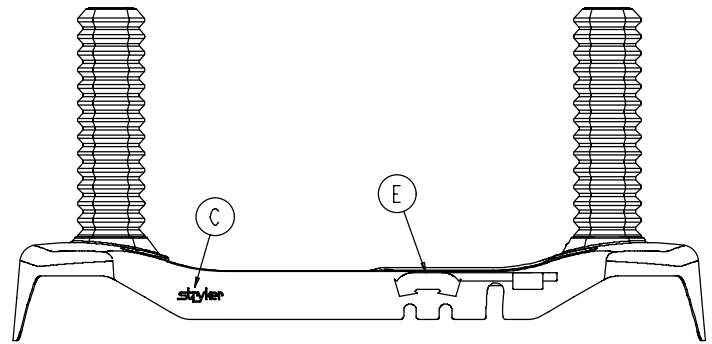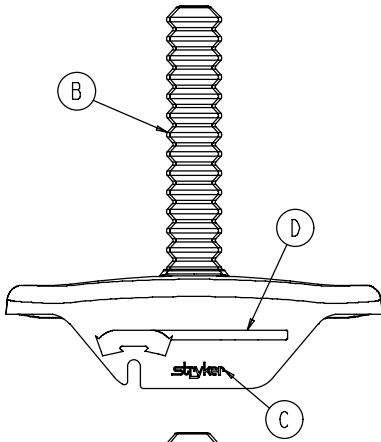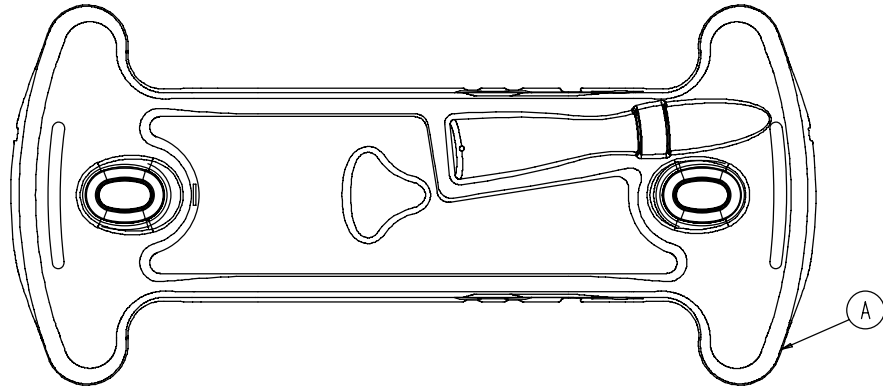

A	0753-010-040	1
B	1040-010-134	2
C	0946-201-060	4


0753-010-063

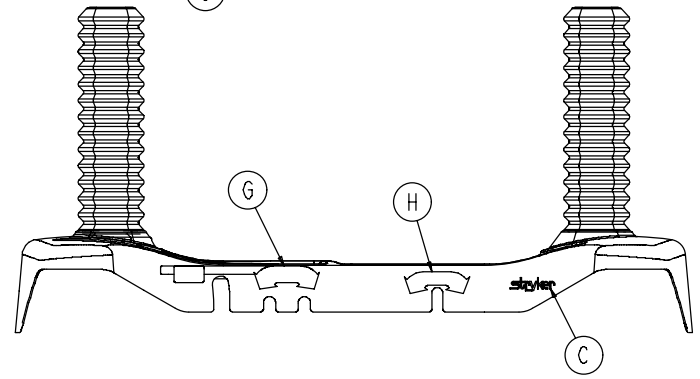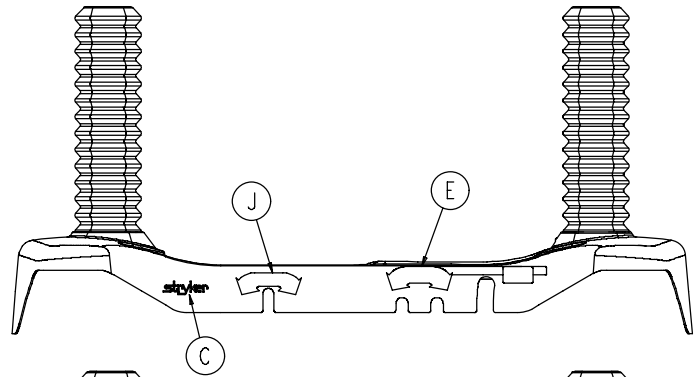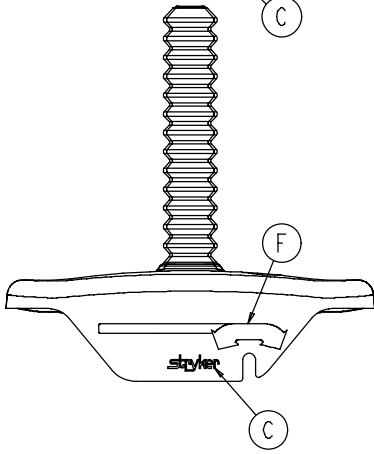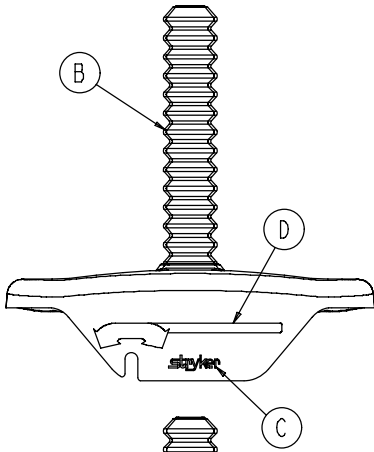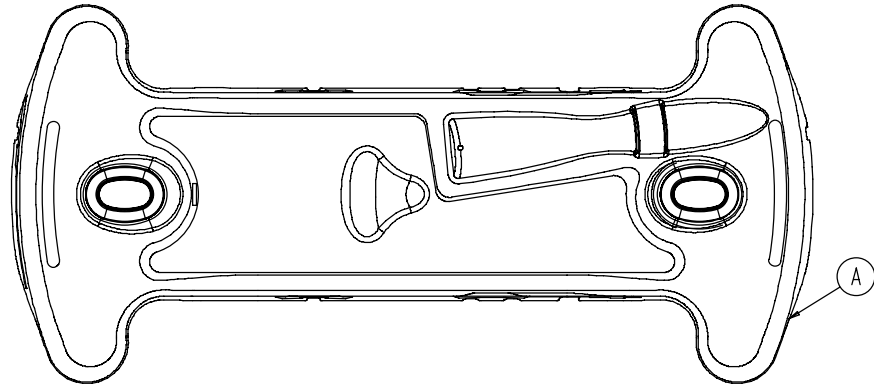


A	0753-010-040	1
B	1040-010-134	2
C	0946-201-060	4
H	0753-010-018	1
J	0753-010-019	1

0753-010-063


D	E	F	G
0753-010-051	0753-010-055	0753-010-017	0753-010-054
0753-010-151	0753-010-155	0753-010-170	0753-010-154
0753-010-251	0753-010-255	0753-010-270	0753-010-254
0753-010-351	0753-010-355	0753-010-370	0753-010-354
0753-010-451	0753-010-455	0753-010-470	0753-010-454
0753-010-551	0753-010-555	0753-010-570	0753-010-554
0753-010-651	0753-010-655	0753-010-670	0753-010-654

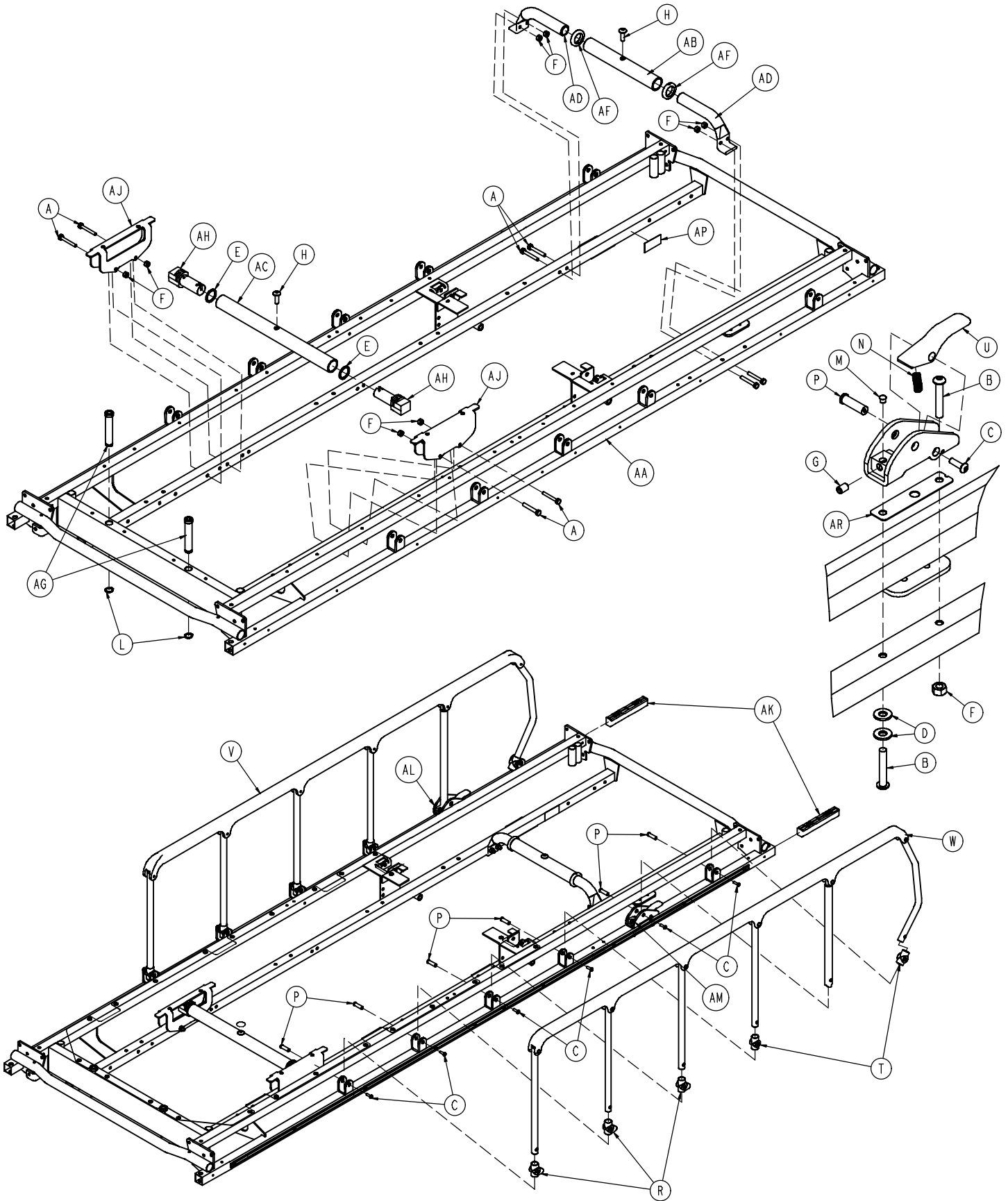

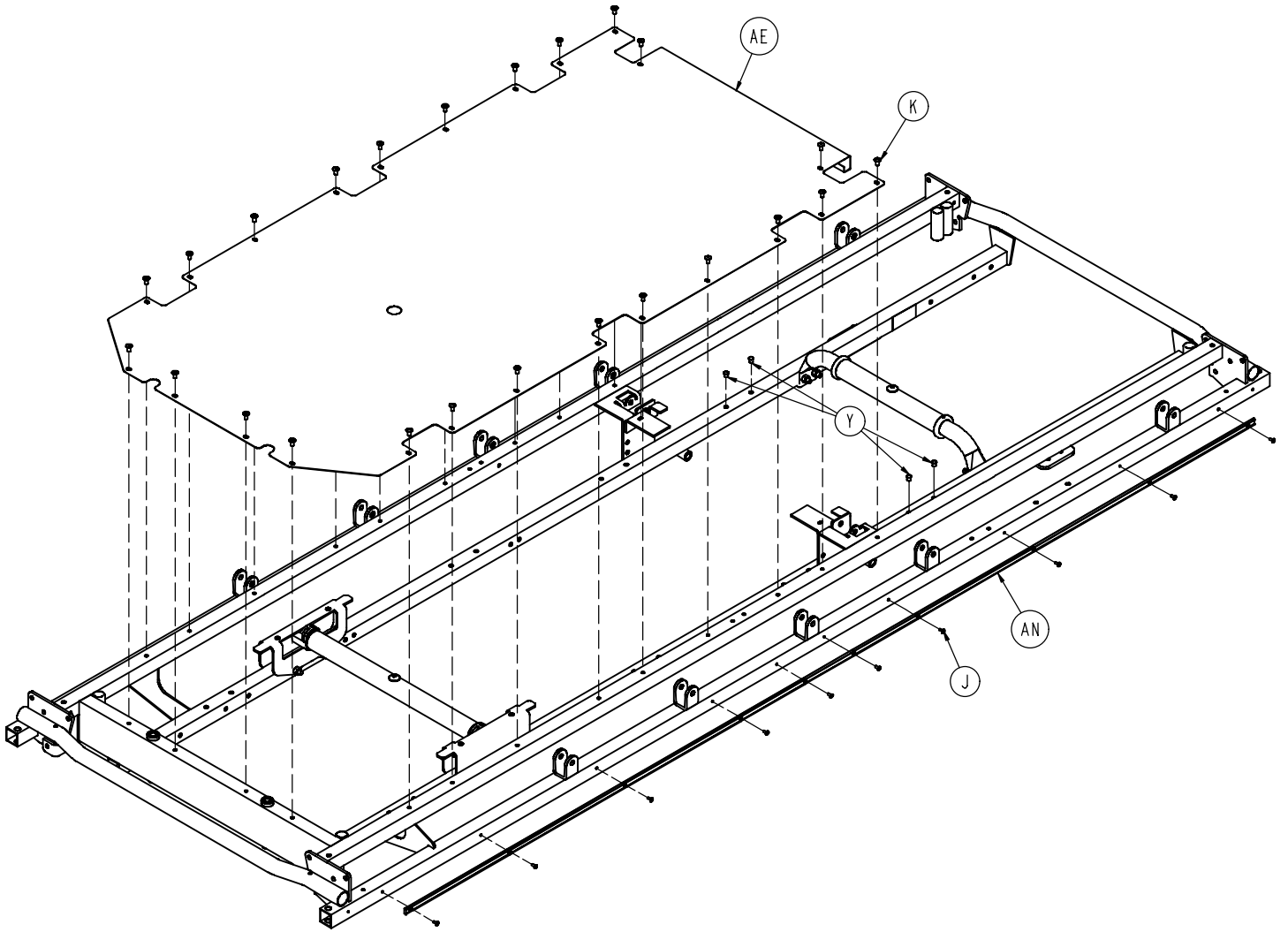


0721-701-010

A	0003-359-000	8
B	0004-135-000	4
C	0004-515-000	14
D	0011-002-000	4
E	0011-360-000	2
F	0016-118-000	10
G	0021-104-000	2
H	0007-074-000	2
J	0025-040-000	20
K	0025-079-000	24
L	0028-023-000	2
M	0037-074-000	6
N	0038-220-000	2
P	0721-026-066	14
R	0721-026-068	6
T	0721-026-069	4
U	0721-026-074	2
V	0721-026-096	1
W	0721-026-097	1
Y	0721-031-065	4
AA	0721-201-020	1
AB	0753-010-036	1
AC	0753-010-042	1
AD	0753-010-043	2
AE	0753-010-044	1
AF	0938-001-401	2
AG	1001-040-012	2
AH	1001-201-029	2
AJ	1001-401-030	2
AK	1010-026-069	2
AL	1010-026-080	1
AM	1010-026-081	1
AN	1010-201-027	2
AP	1210-800-008	1
AR	1010-026-103	2

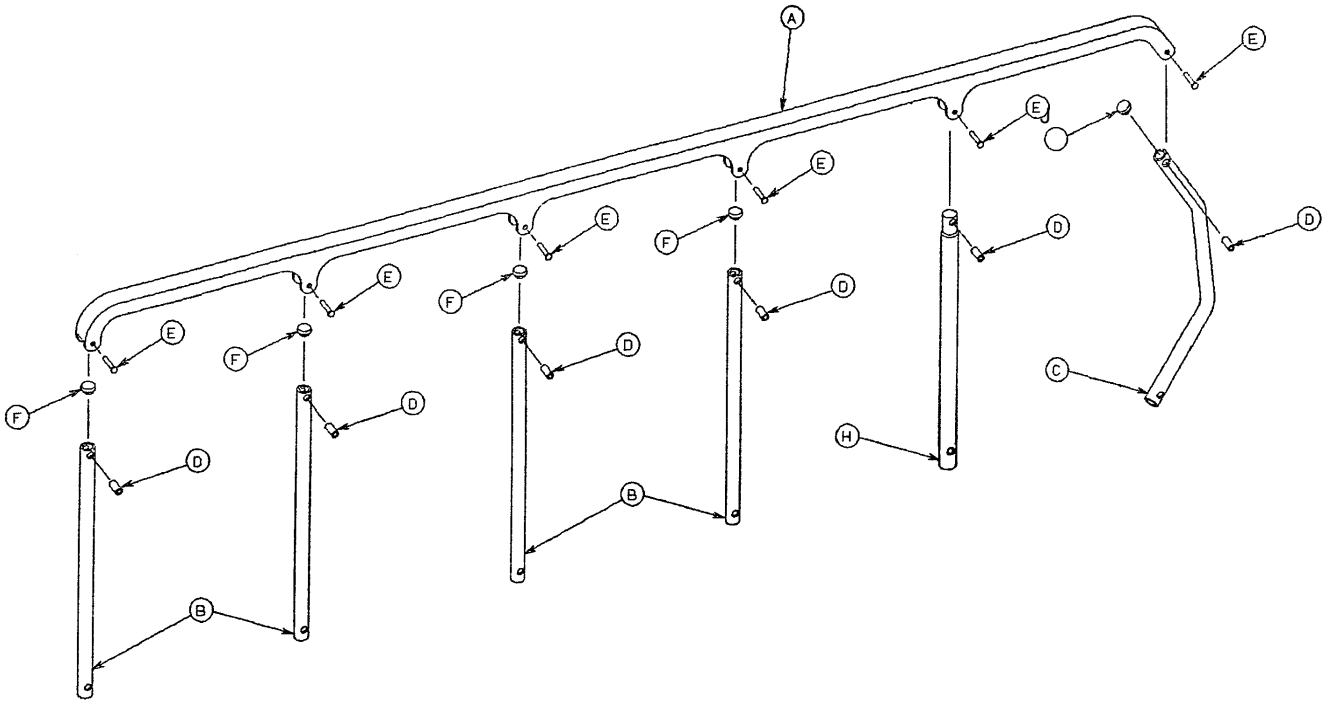


A	1010-026-015	1
B	1010-026-083	4
C	1010-026-084	1
D	1010-026-082	6
E	0025-106-000	6
F	1010-026-010	4
H	1010-026-085	1
J	1010-026-012	1

0721-231-110

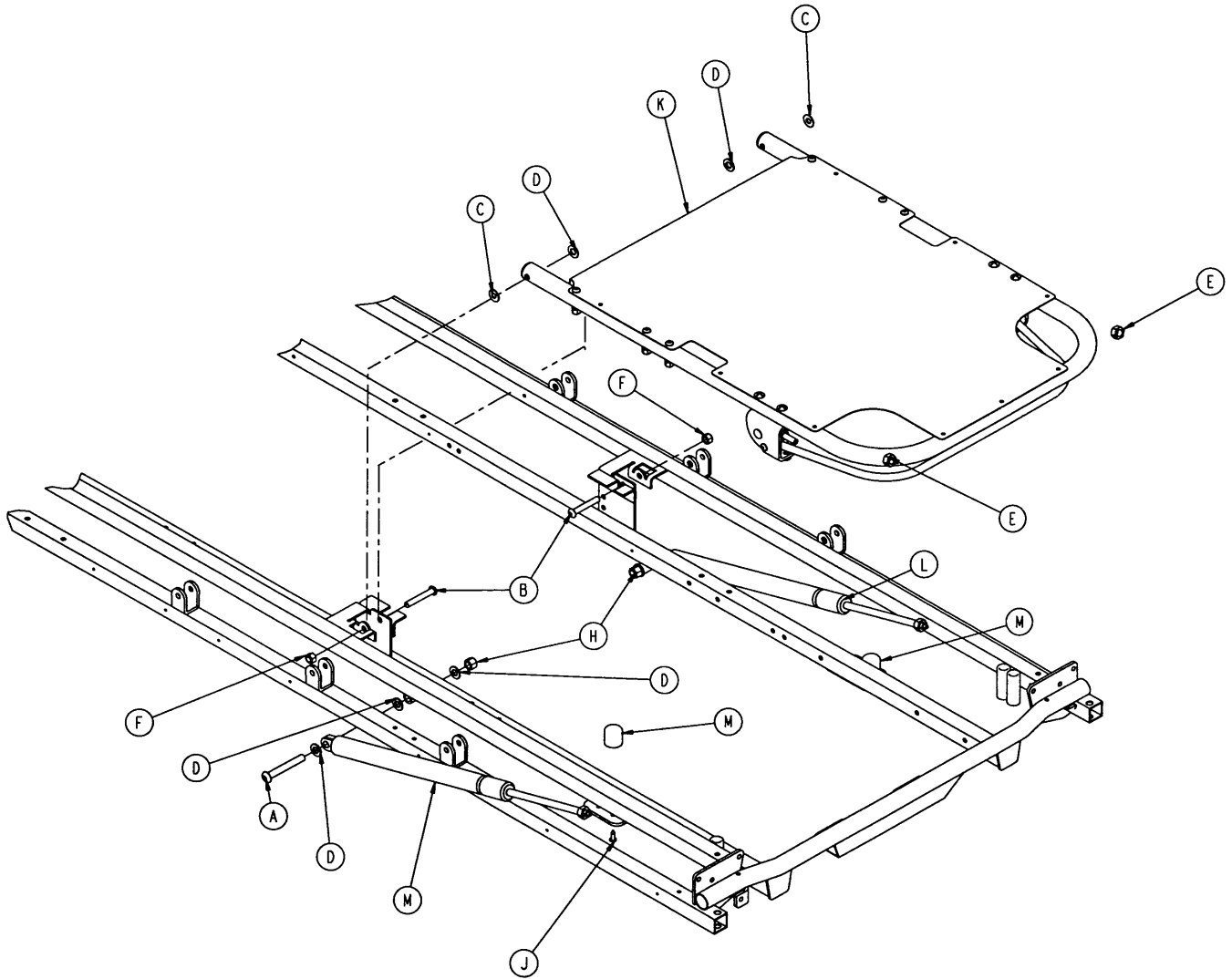


A	0004-182-000	2
B	0004-183-000	2
C	0011-179-000	2
D	0014-021-000	8
E	0015-059-000	2
F	0016-116-000	2
H	0016-117-000	2
J	0023-100-000	2
K	0721-601-042	1
L	1010-031-078	2
M	1211-031-031	2

0721-231-120

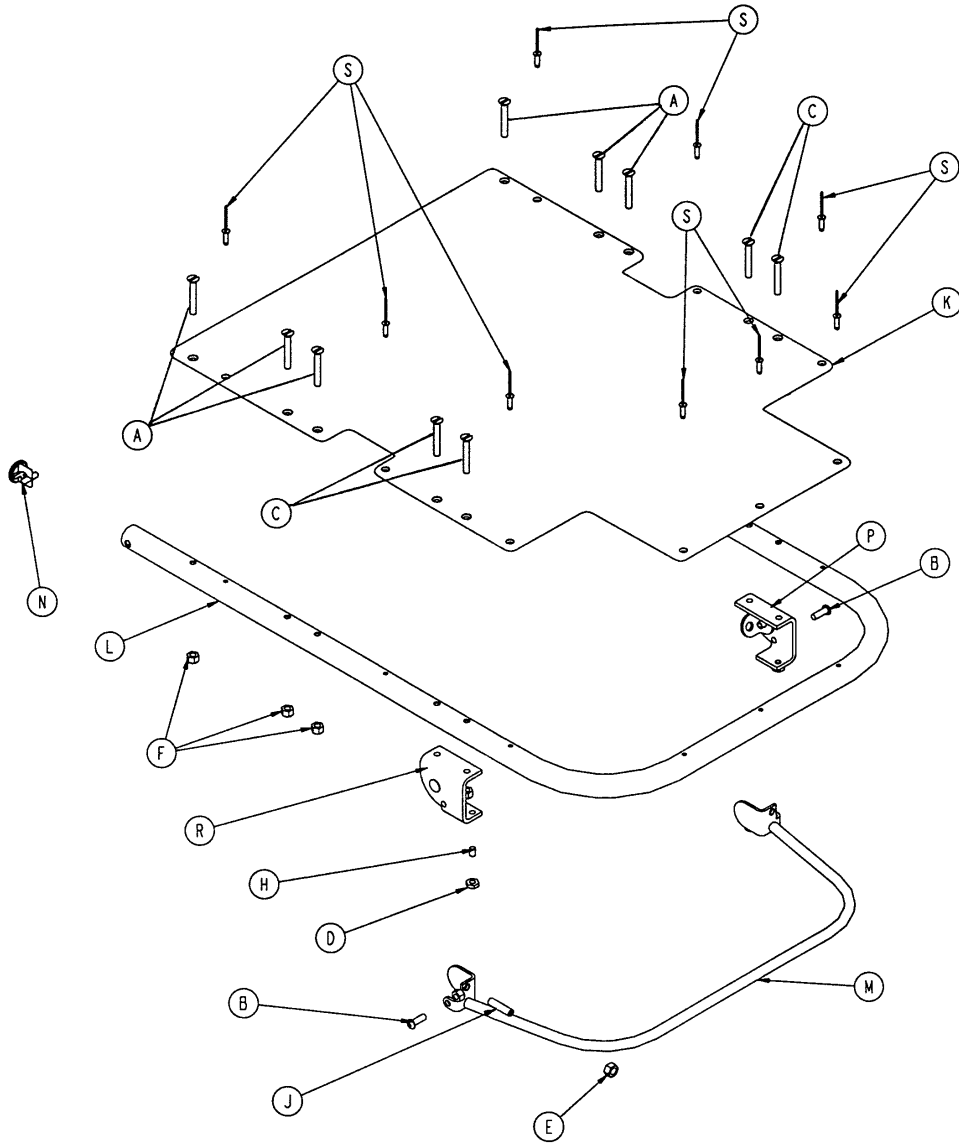


A	0001-096-000	6
B	0004-161-000	2
C	0007-071-000	4
D	0015-037-000	2
E	0015-050-000	2
F	0016-118-000	12
H	0021-125-000	2
J	0021-126-000	2
K	0721-601-035	1
L	0721-231-013	1
M	0721-231-118	1
N	1001-001-036	2
P	1210-031-106	1
R	1210-031-107	1
S	0025-142-000	9

1210-031-106 & 1210-031-107

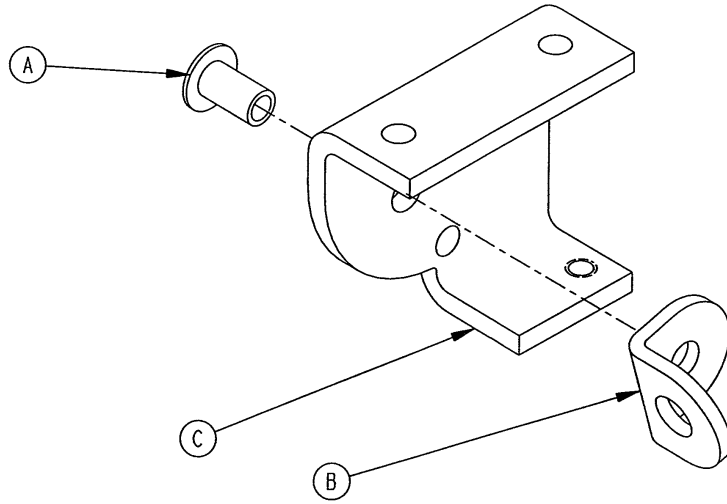


A	0025-144-000	1
B	1210-031-103	1
C	1210-031-104	1

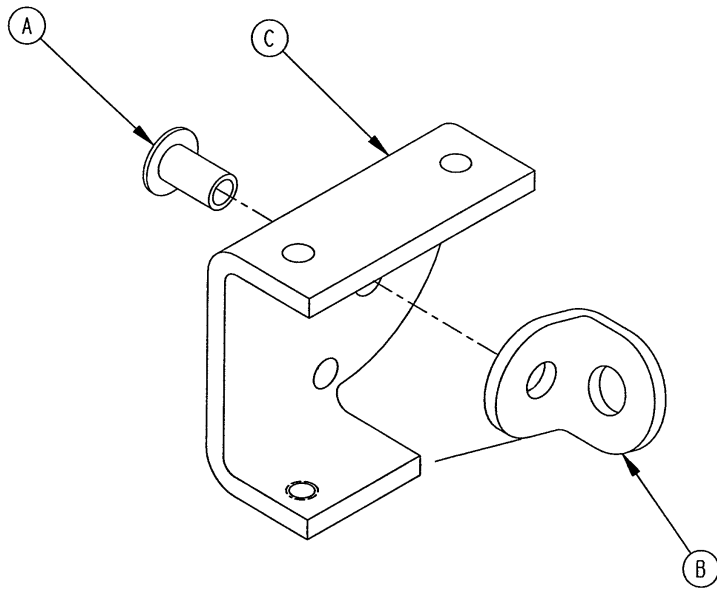


A	0025-144-000	1
B	1210-031-103	1
C	1210-031-105	1

0721-600-010

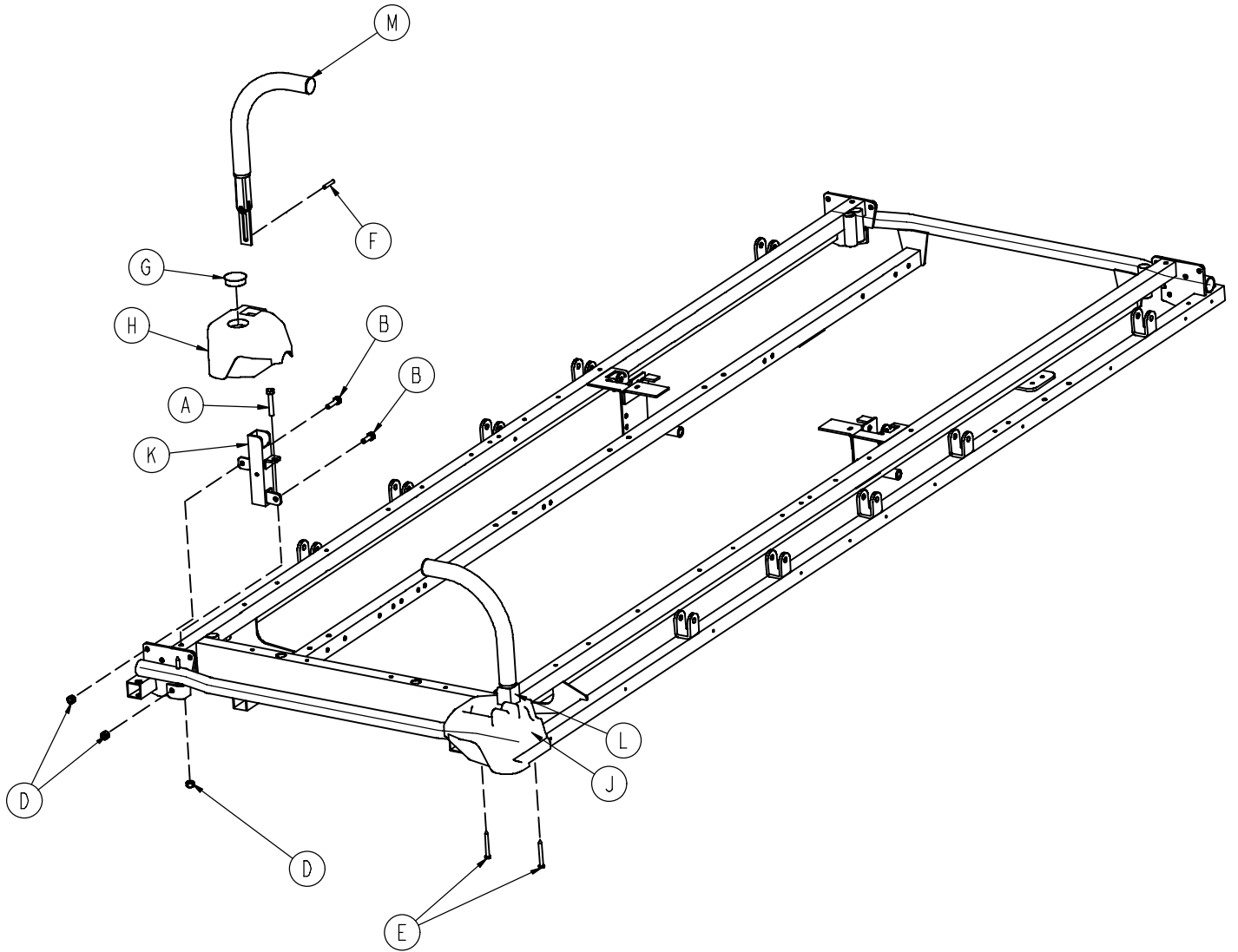


A	0003-355-000	2
B	0003-356-000	4
D	0016-118-000	6
E	0023-283-000	4
F	0026-114-000	2
G	0037-059-000	1
H	1010-201-236	1
J	1010-201-237	1
K	1010-254-004	1
K	1010-254-006	1
M	1211-351-010	2


1211-351-010

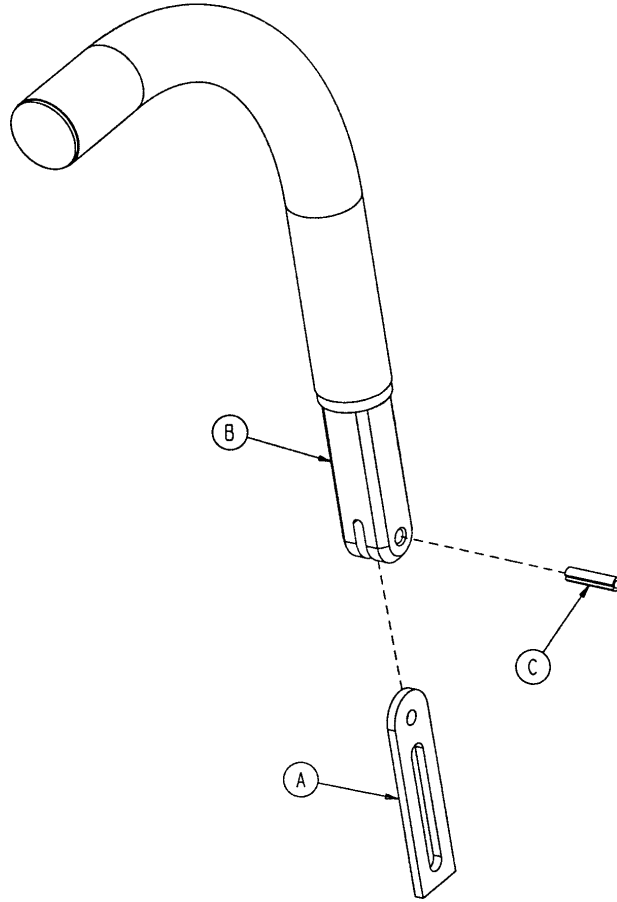

A	0026-118-000	1
B	1010-354-024	1
C	1211-151-018	1


A	0024-023-000	1
B	0390-003-053	1
C	0393-003-043	1
D	0004-496-000	1

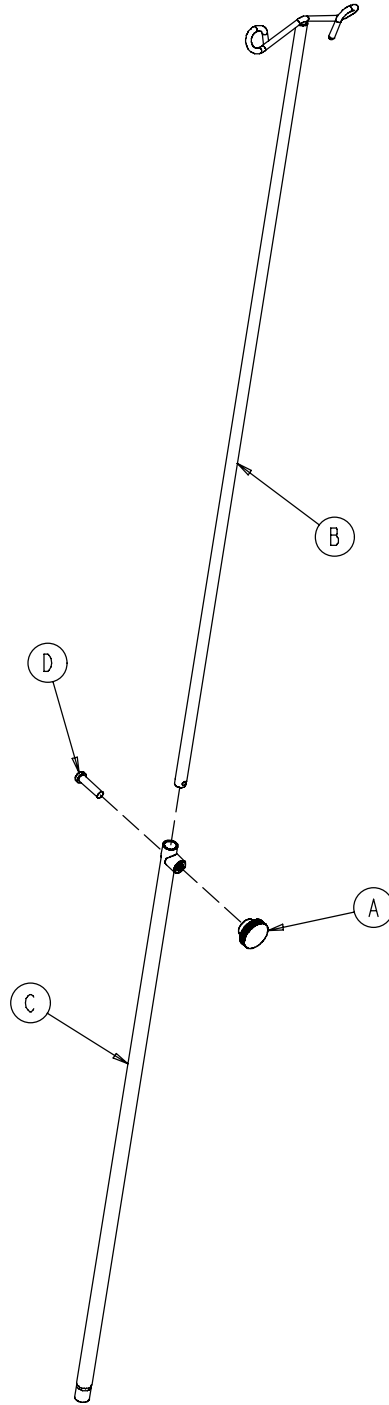

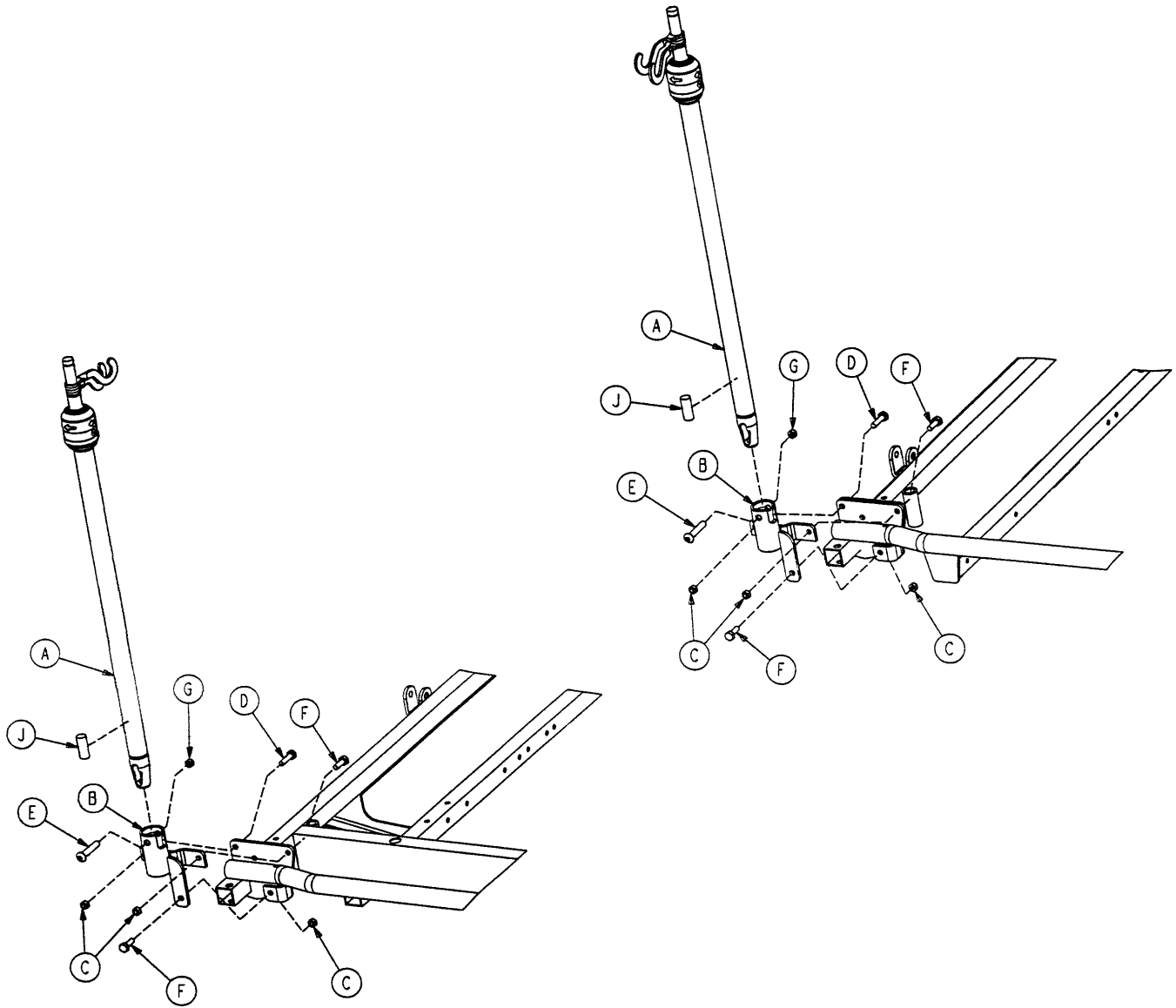


A	1211-210-010	1
B	1001-259-103	1
C	0016-118-000	3
D	0003-357-000	1
E	0004-452-000	1
F	0003-356-000	2
G	0016-116-000	1
J	1001-059-040	1

1211-210-010

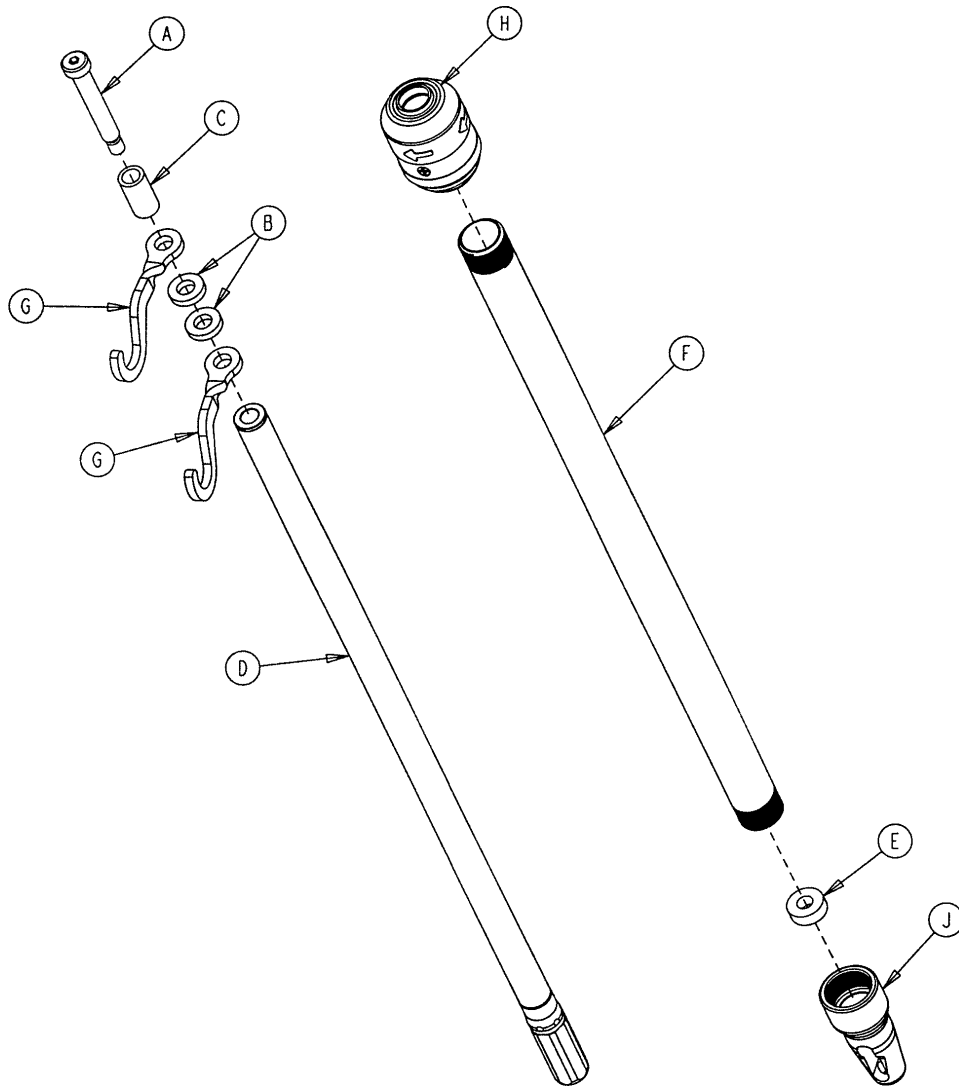


A	0008-031-000	1
B	0052-017-000	2
C	0926-400-162	1
D	1211-210-029	1
E	1001-359-013	1
F	1001-159-028	1
G	1010-059-016	2
H	1211-110-026	1
J	1001-359-112	1

1211-210-026

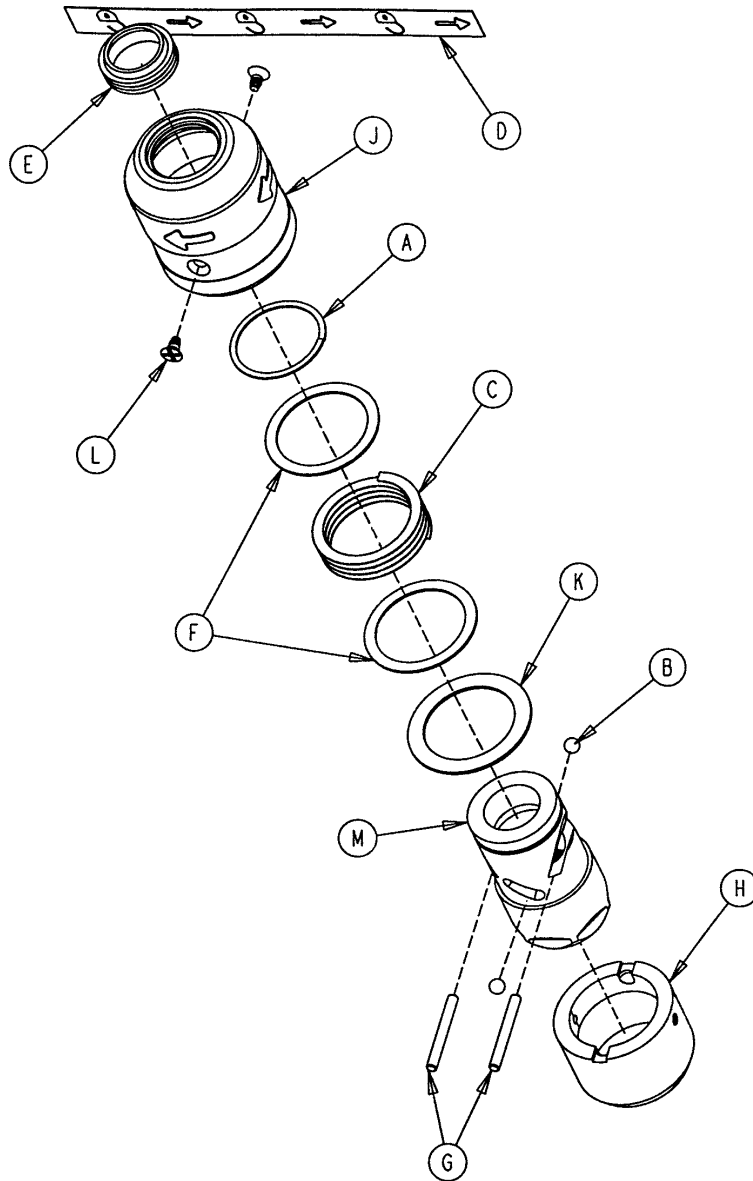


A	0028-167-000	1
B	0031-004-000	2
C	0038-392-000	1
D	1211-091-034	1
E	1211-110-018	1
F	1211-110-020	2
G	1211-110-021	2
H	1211-110-022	1
J	1211-110-024	1
K	1211-110-035	1
L	1211-110-036	2
M	1211-210-023	1

0721-061-000 & 0721-062-000

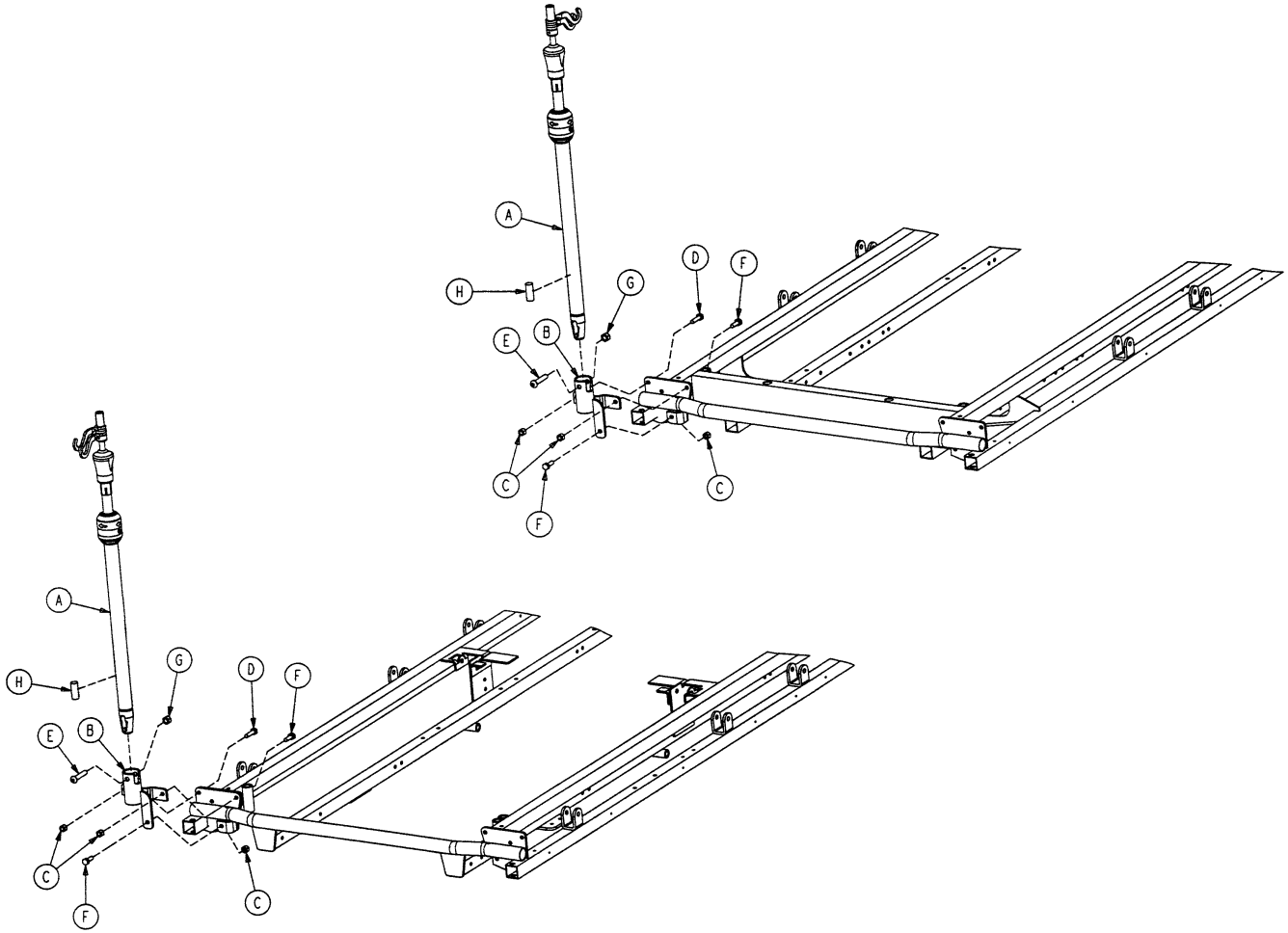


A	1211-211-010	1
B	1001-259-013	1
C	0016-118-000	3
D	0003-357-000	1
E	0004-452-000	1
F	0003-356-000	2
G	0016-116-000	1
H	1001-059-040	1

1211-211-010

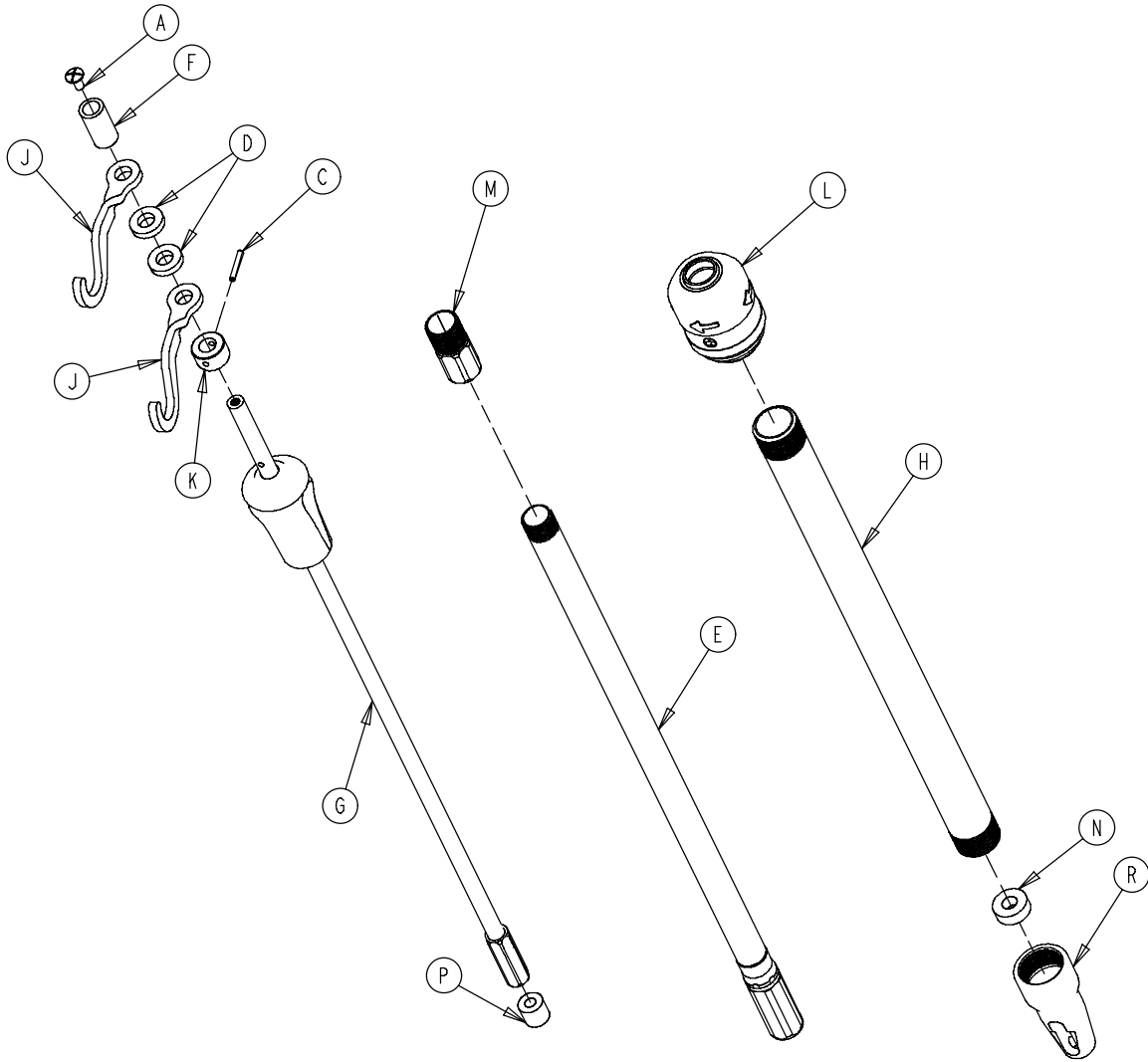


A	0007-004-000	1
C	0026-076-000	1
D	0052-017-000	2
E	1211-210-031	1
F	0926-400-162	1
G	1211-110-032	1
H	1001-161-023	1
J	1010-059-016	2
K	1010-061-014	1
L	1211-210-026	1
M	1211-110-016	1
N	1001-359-013	1
P	1001-259-014	1
R	1001-359-112	1

1211-110-032

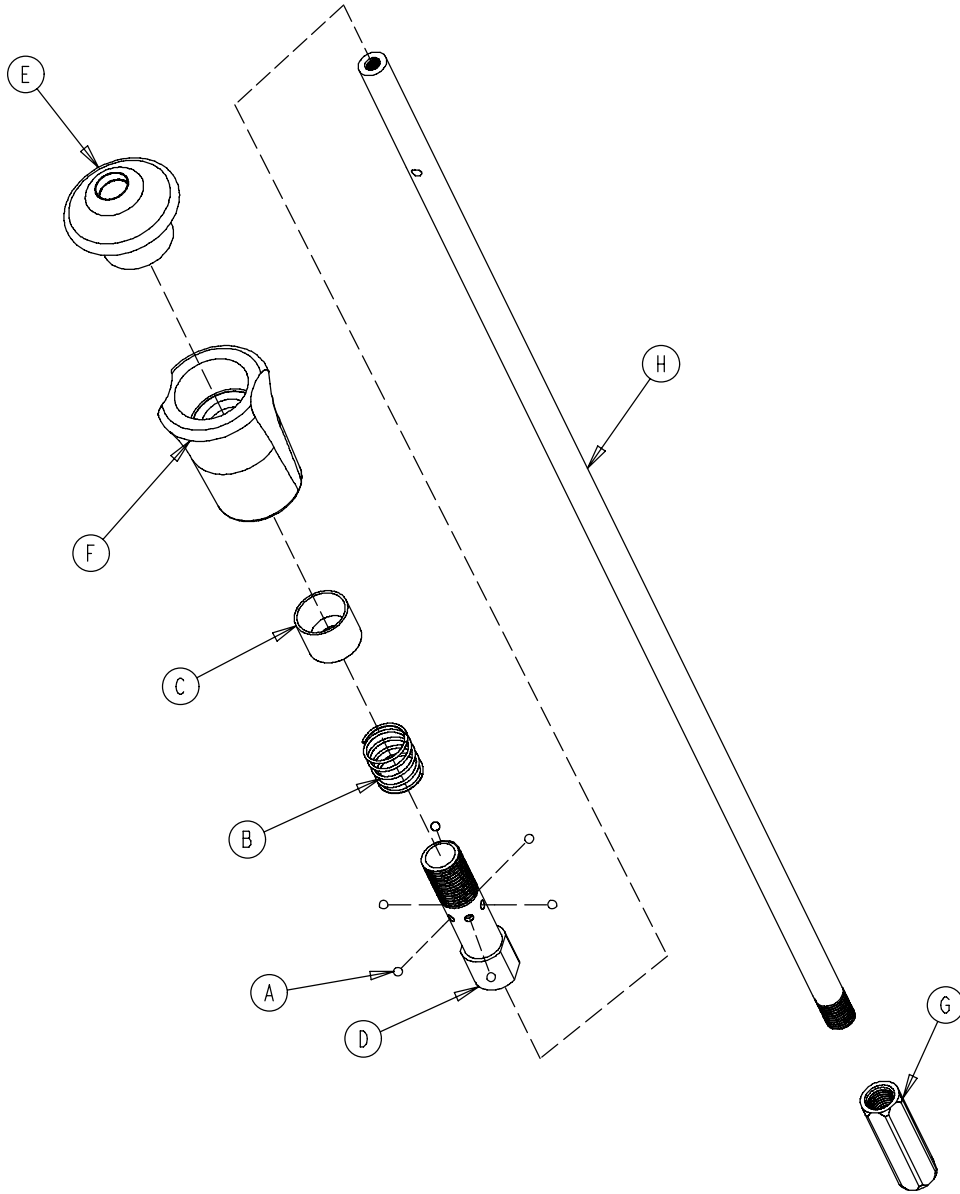


A	0031-021-000	6
B	0038-303-000	1
C	1010-061-013	1
D	1010-061-016	1
E	1010-061-017	1
F	1010-061-018	1
G	1211-110-117	1
H	1211-110-033	1

0785-045-200

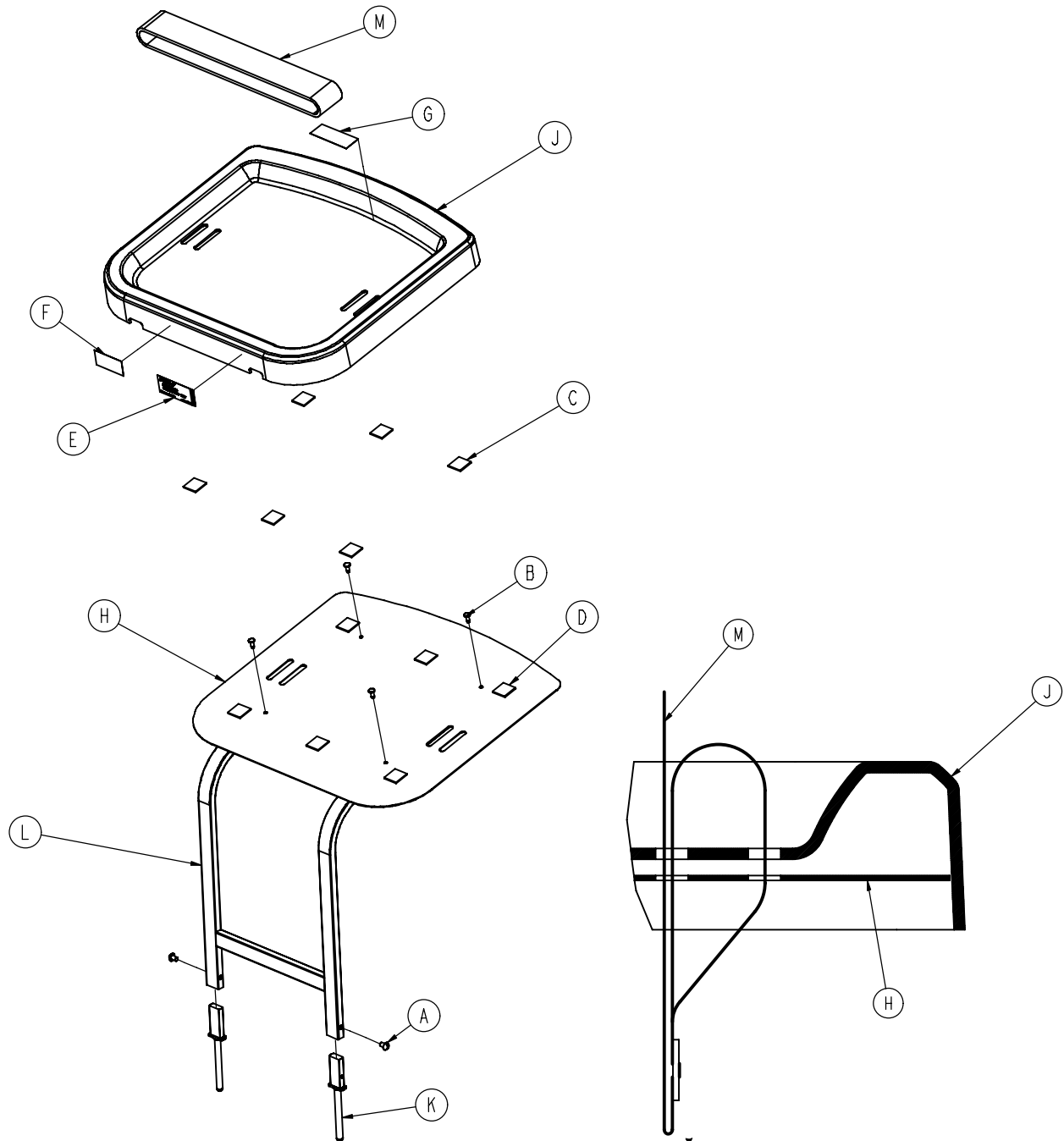


A	0002-0044-000	2	G	0785-045-403	1
B	0025-055-000	4	H	0785-045-204	1
C	0029-008-000	6	J	0785-045-207	1
D	0029-010-000	6	K	0785-045-208	2
E	0785-045-201	1	L	0785-045-210	1
F	0785-045-402	1	M	1010-050-021	1

0785-045-400

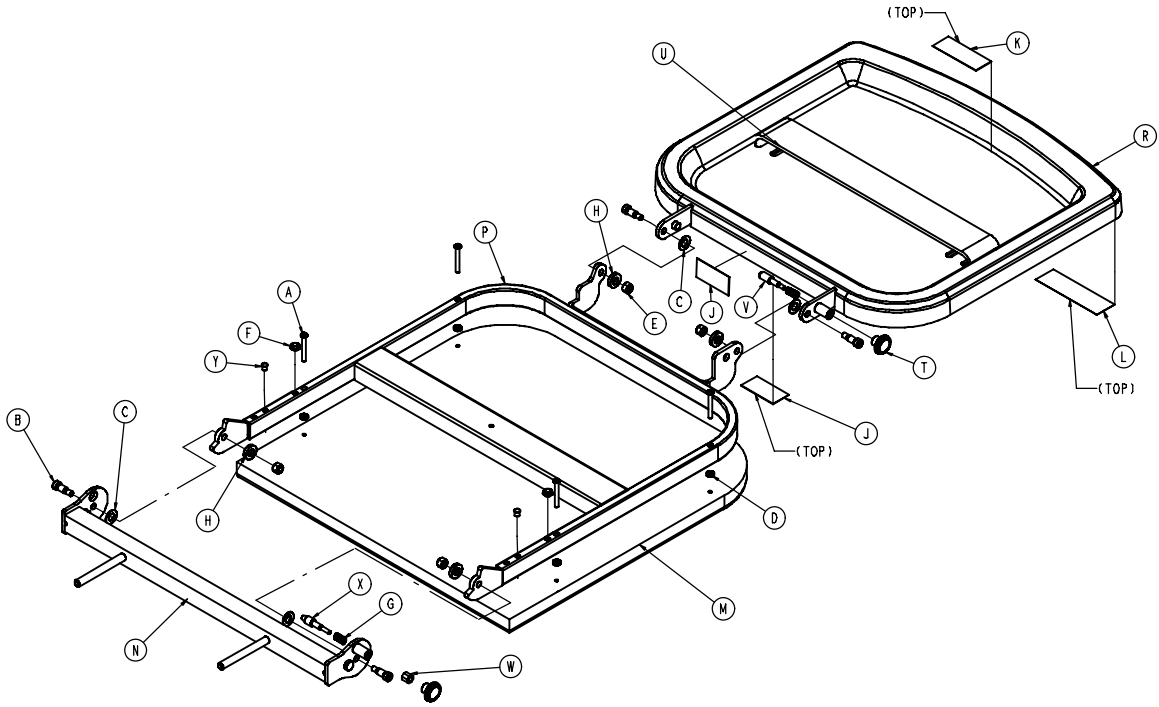


A	0004-215-000	4	M	0785-045-405	1
B	0008-049-000	4	N	0785-045-410	1
C	0014-020-000	4	P	0785-045-420	1
D	0014-120-000	4	R	0785-045-430	1
E	0016-028-000	4	T	0785-045-450	2
F	0037-052-000	2	U	1010-050-021	1
G	0038-133-000	2	V	1010-050-242	1
H	0052-017-000	4	W	1010-050-250	1
J	0785-045-402	2	X	1010-050-248	1
K	0785-045-403	1	Y	0721-031-065	2
L	0785-045-401	1			

0785-045-500

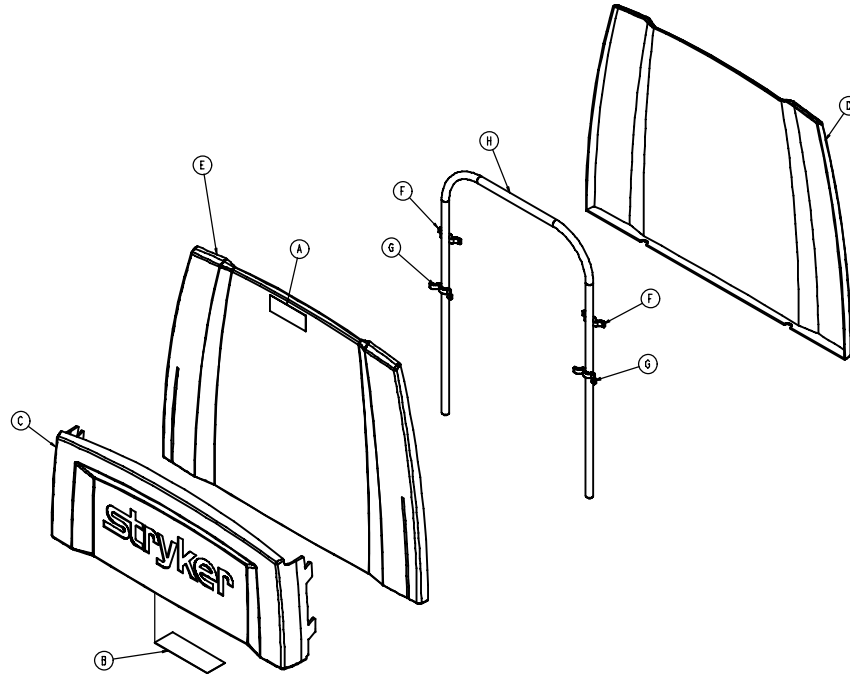


A	0785-045-402	1	E	0785-045-513	1
B	0785-045-501	1	F	0785-045-514	2
C	0785-045-511	1	G	0785-045-515	2
D	0785-045-512	1	H	0946-029-010	1

1020-130-000

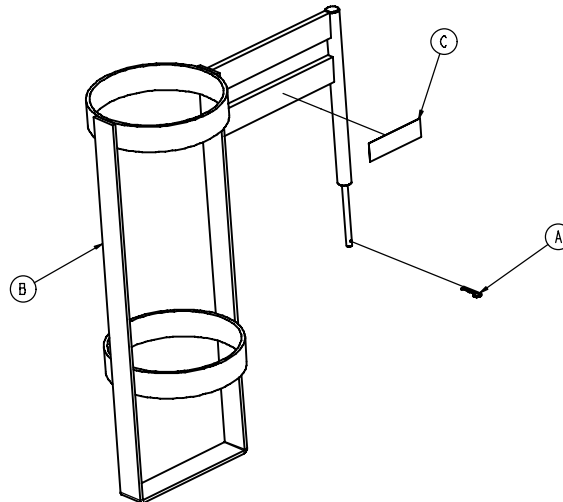

A	0027-030-000	1
B	1020-030-111	1
C	1020-030-017	1

1040-710-095

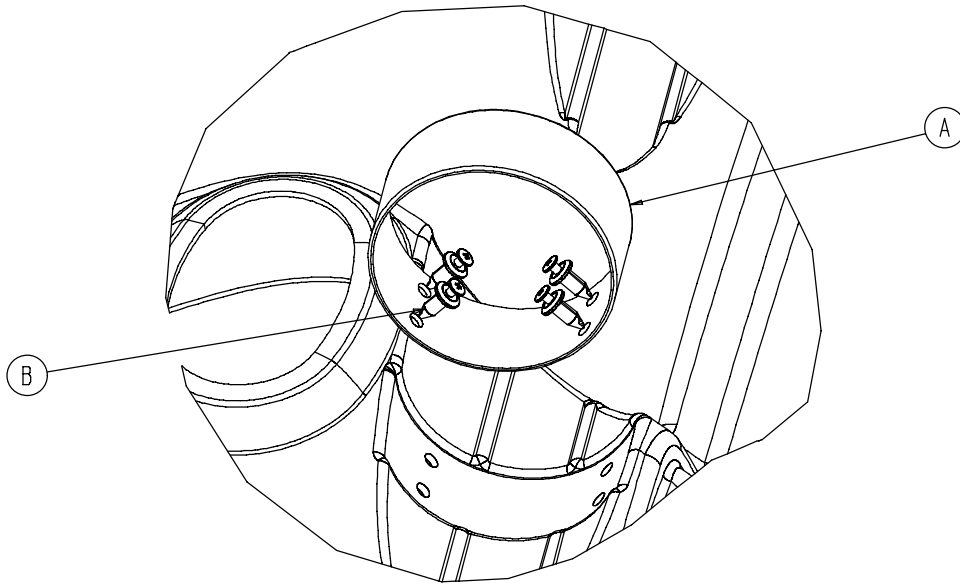

A	1040-010-091	1
B	1040-010-092	4

European Representative

Stryker EMEA RA/QA Director
Stryker France
ZAC Satolas Green Pusignan
Av. De Satolas Green
69881 MEYZIEU Cedex
France

stryker[®]

3800 E. Centre Ave., Portage, MI 49002
(800) 327-0770
www.stryker.com



JH 10/06 0735-600-002 REV D