


Parts Manual


Medical

**M-Series Stretcher
Model SM204**

**For parts or technical
assistance call
800 327 0770 (option 2)**




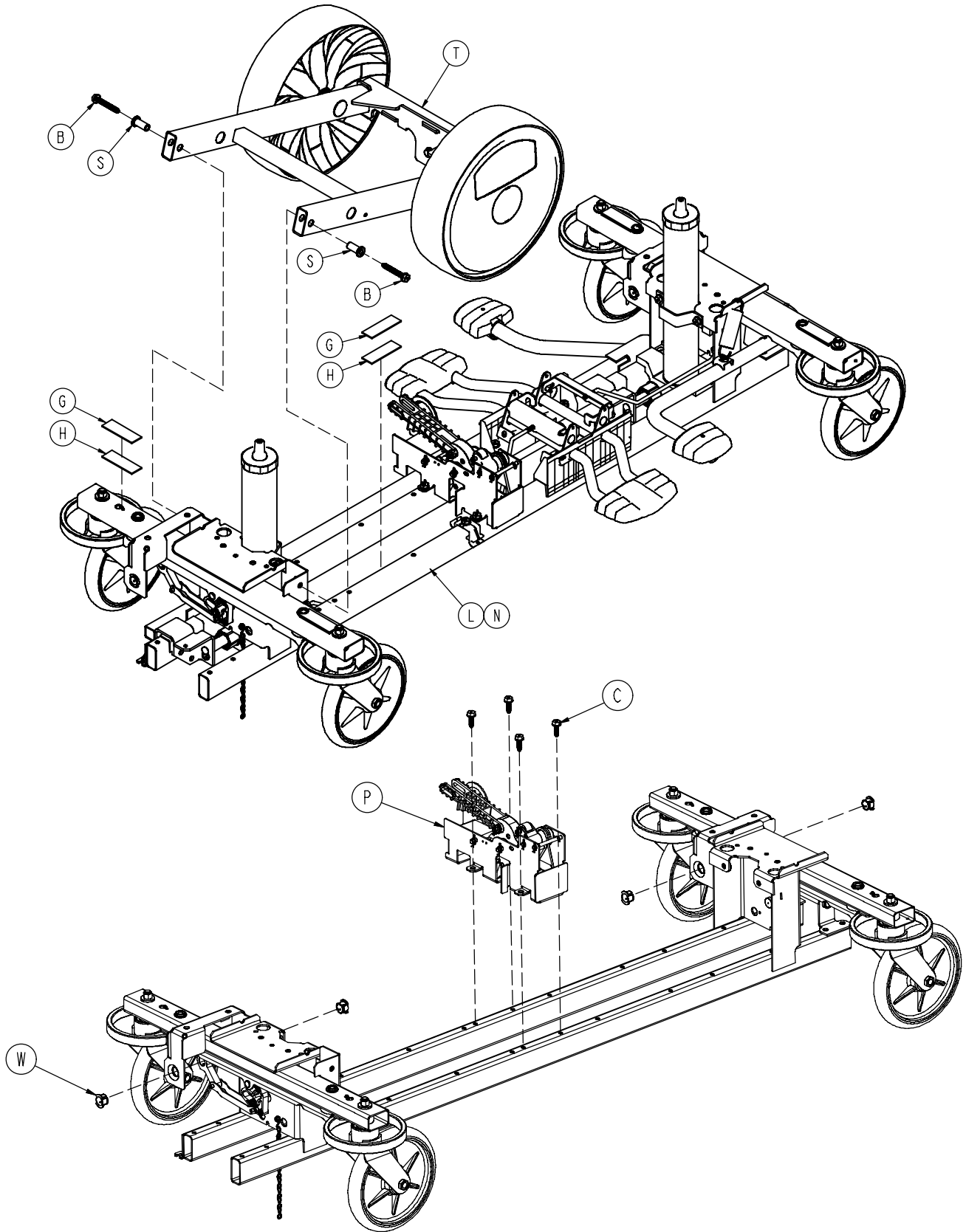

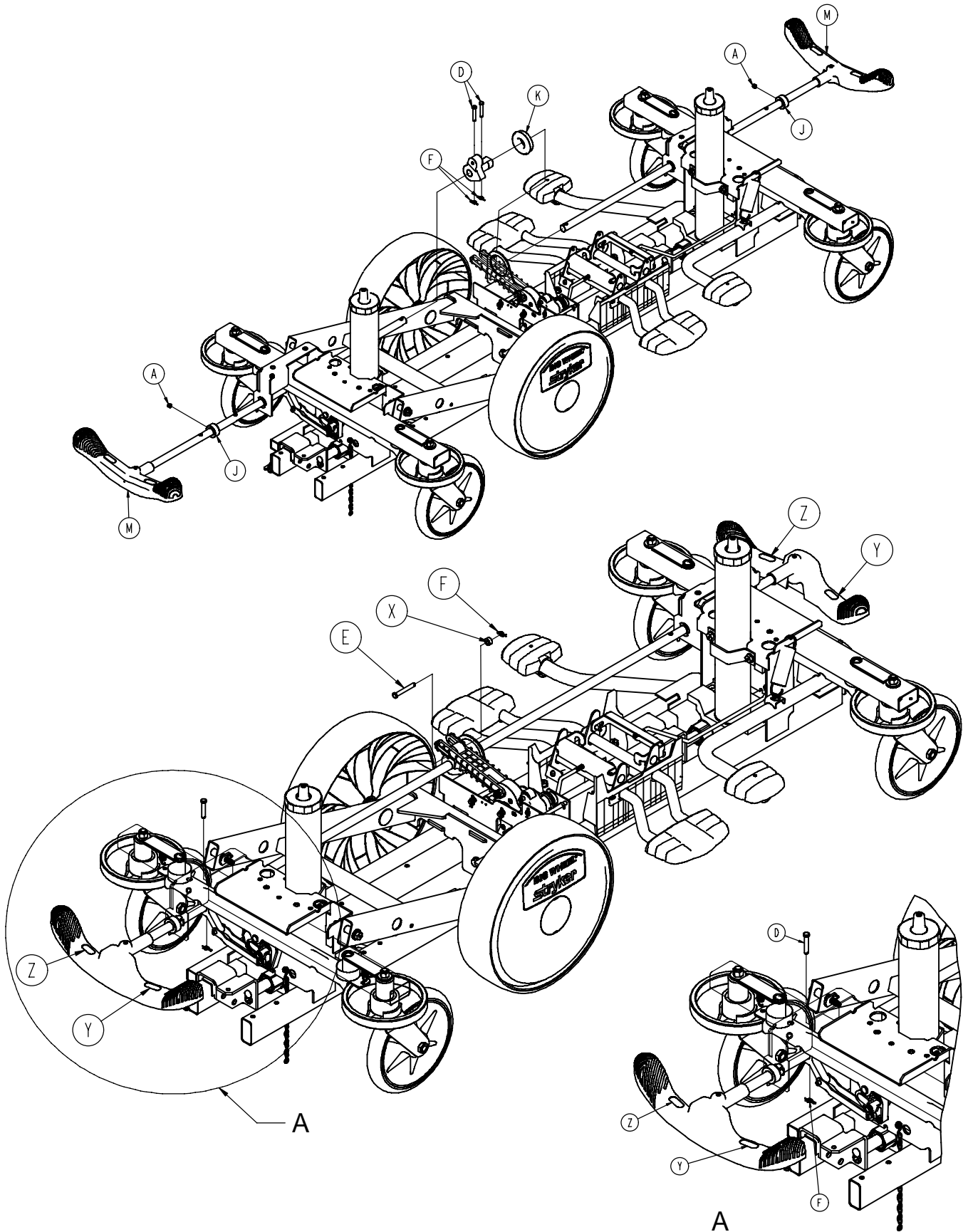


1040-015-305

A	0021-166-000	2
B	0023-150-000	2
C	0023-288-000	4
D	0026-273-000	4
E	0026-330-000	1
F	0027-020-000	5
G	0029-007-000	5
H	0029-009-000	5
J	0042-020-000	2
K	0081-330-000	1
L	1040-003-205	1
M	1040-003-291	2
N	1040-005-500	1
P	1040-006-201	1
R	1040-006-231	1
S	1040-015-007	2
T	1040-015-201	1
W	1210-201-251	4
X	0052-015-000	1
Y	1210-201-335	2
Z	1210-201-336	2

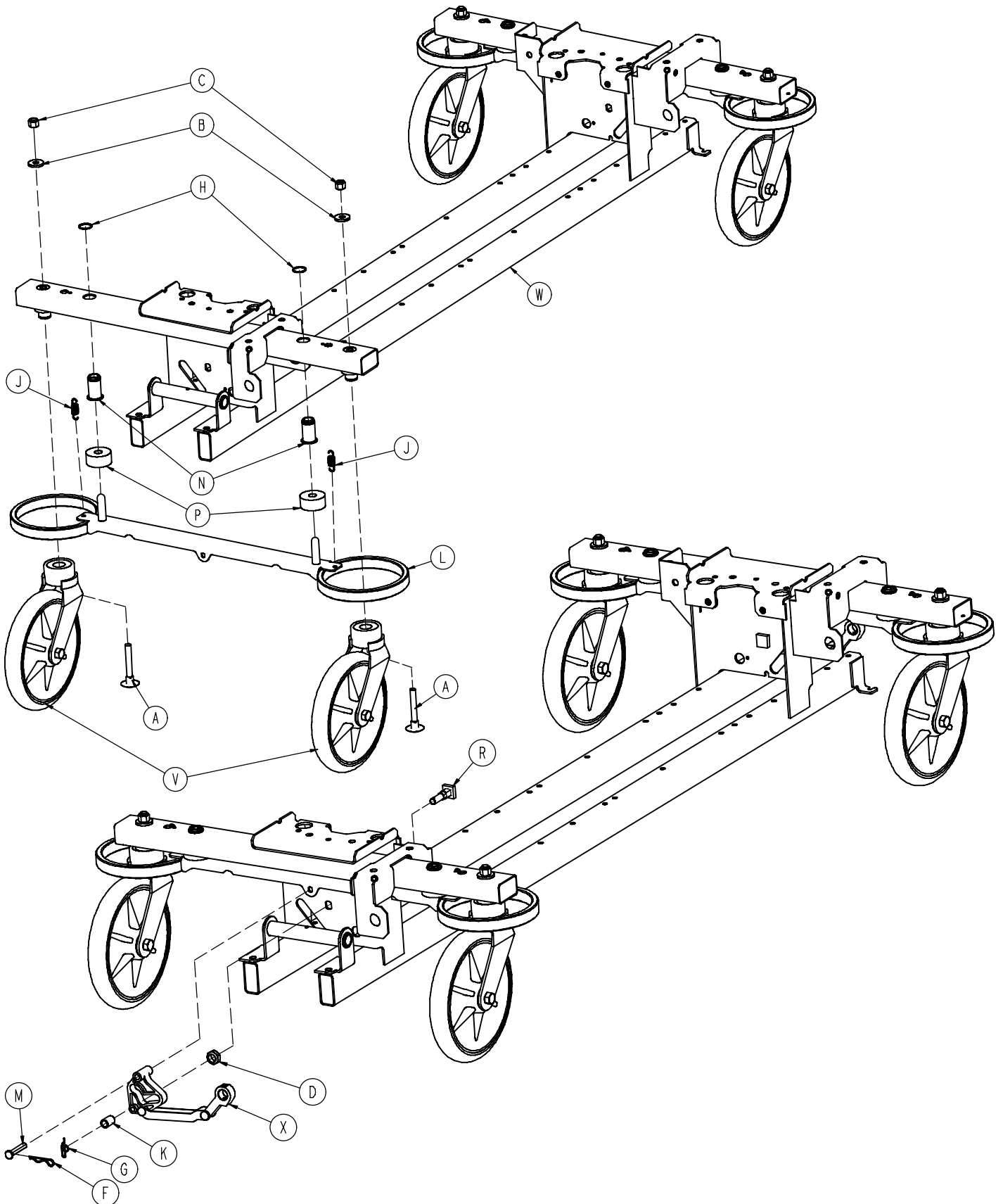


1040-003-205

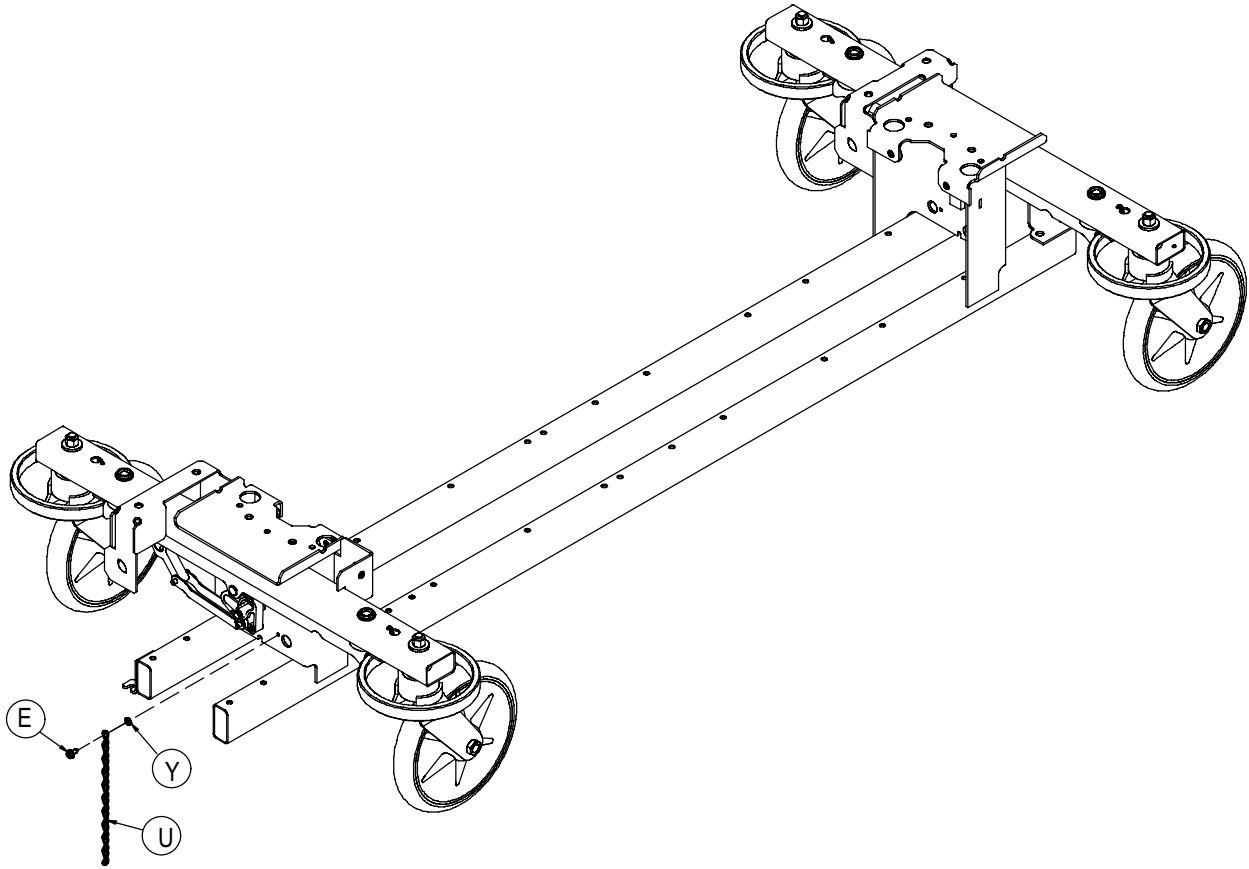


A	0005-044-000	4
B	0011-262-000	4
C	0016-035-000	4
D	0016-049-000	2
E	0003-364-000	1
F	0027-012-000	2
G	0027-022-000	2
H	0028-037-000	4
J	0038-439-000	4
K	0081-037-000	2
L	0753-003-128	2
M	0753-003-066	2
N	0753-003-079	4
P	0753-003-121	4
R	0753-003-230	2
U	3001-200-052	1
V	0753-010-020	4
W	1040-001-301	1
X	1040-003-185	2
Y	0013-018-000	1

0753-010-220

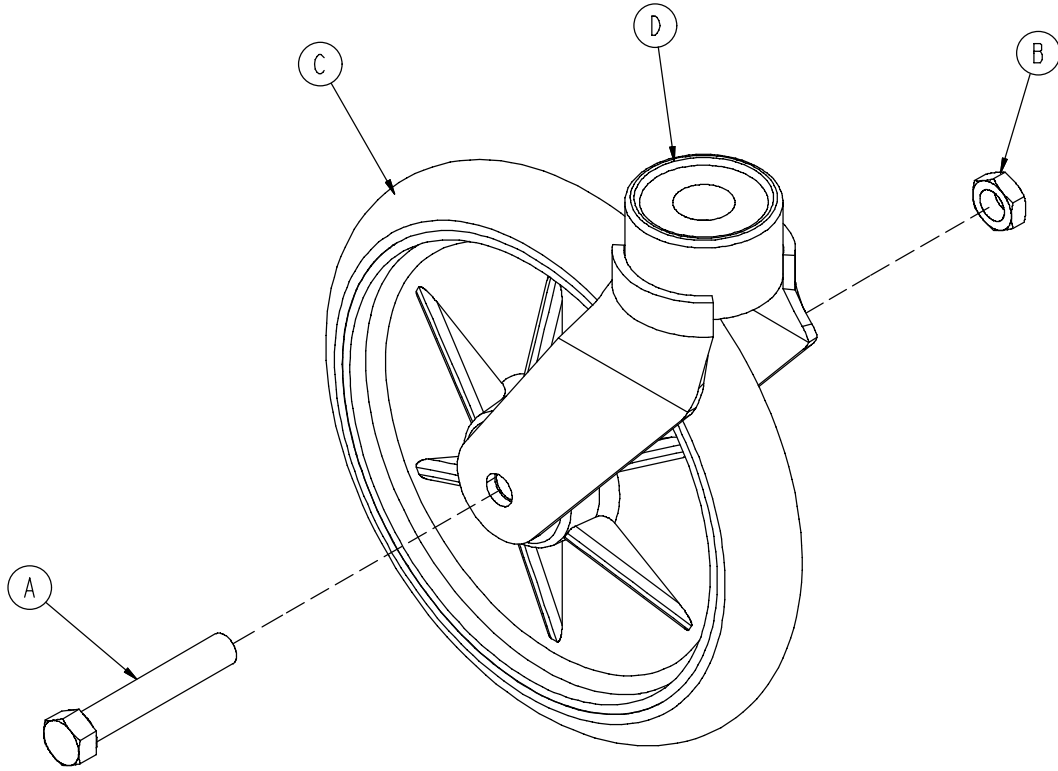


A	0003-099-000	1
B	0016-060-000	1
C	0753-003-215	1
D	0753-010-021	1

1040-700-009

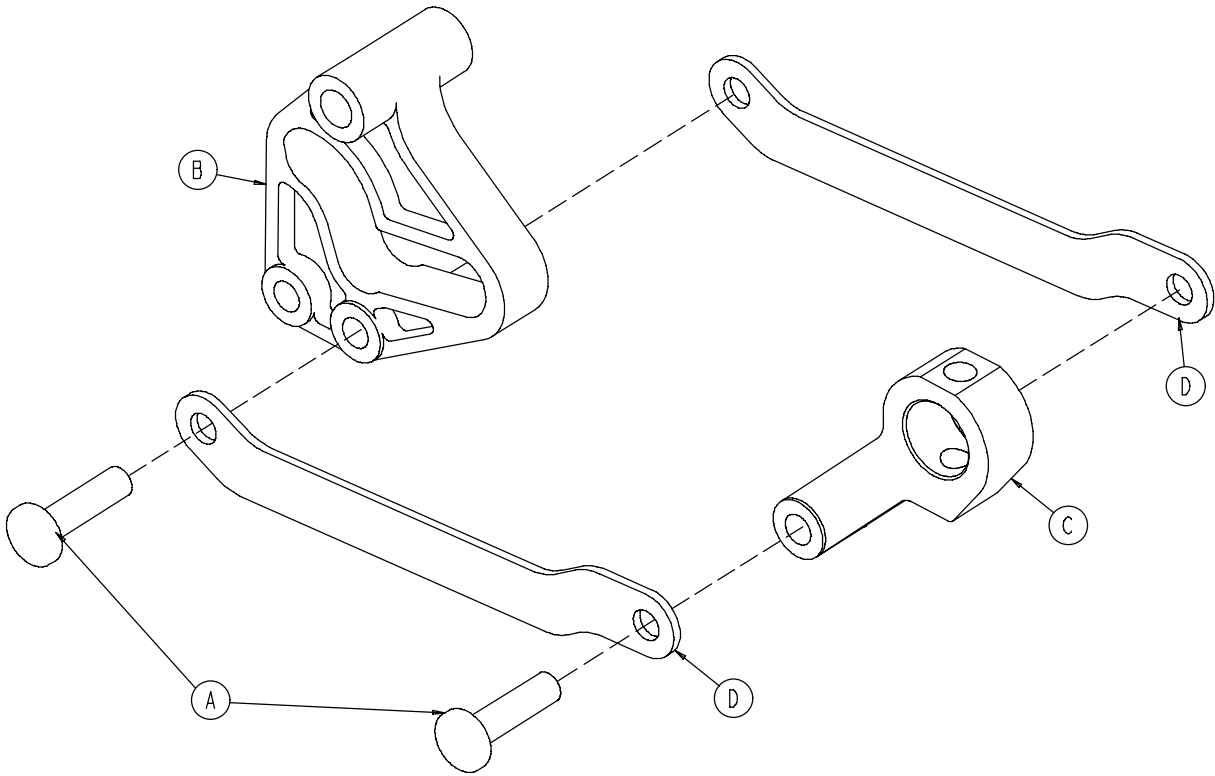




A	0753-003-098	2
B	0753-203-102	1
C	1040-003-174	1
D	1040-003-076	2

1040-003-291



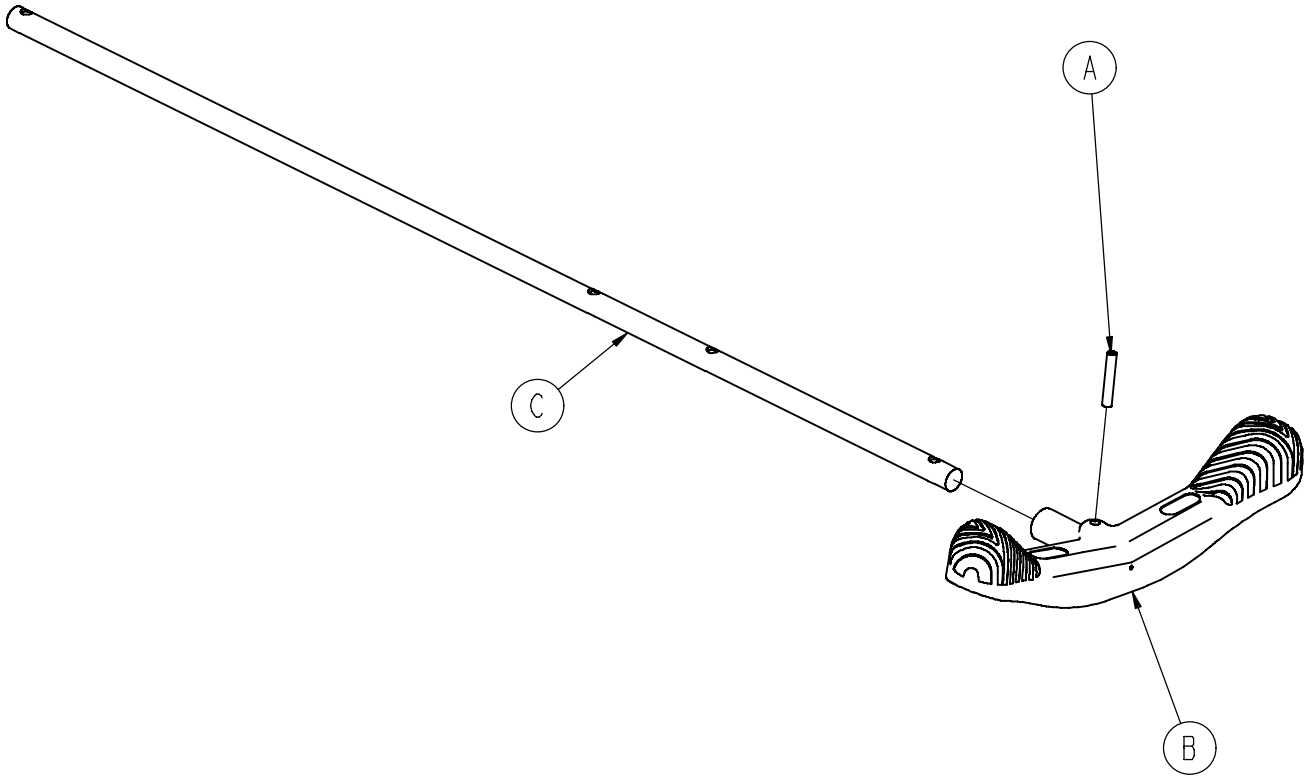


A	0026-067-000	1
B	1040-005-154	1
C	1040-003-113	1

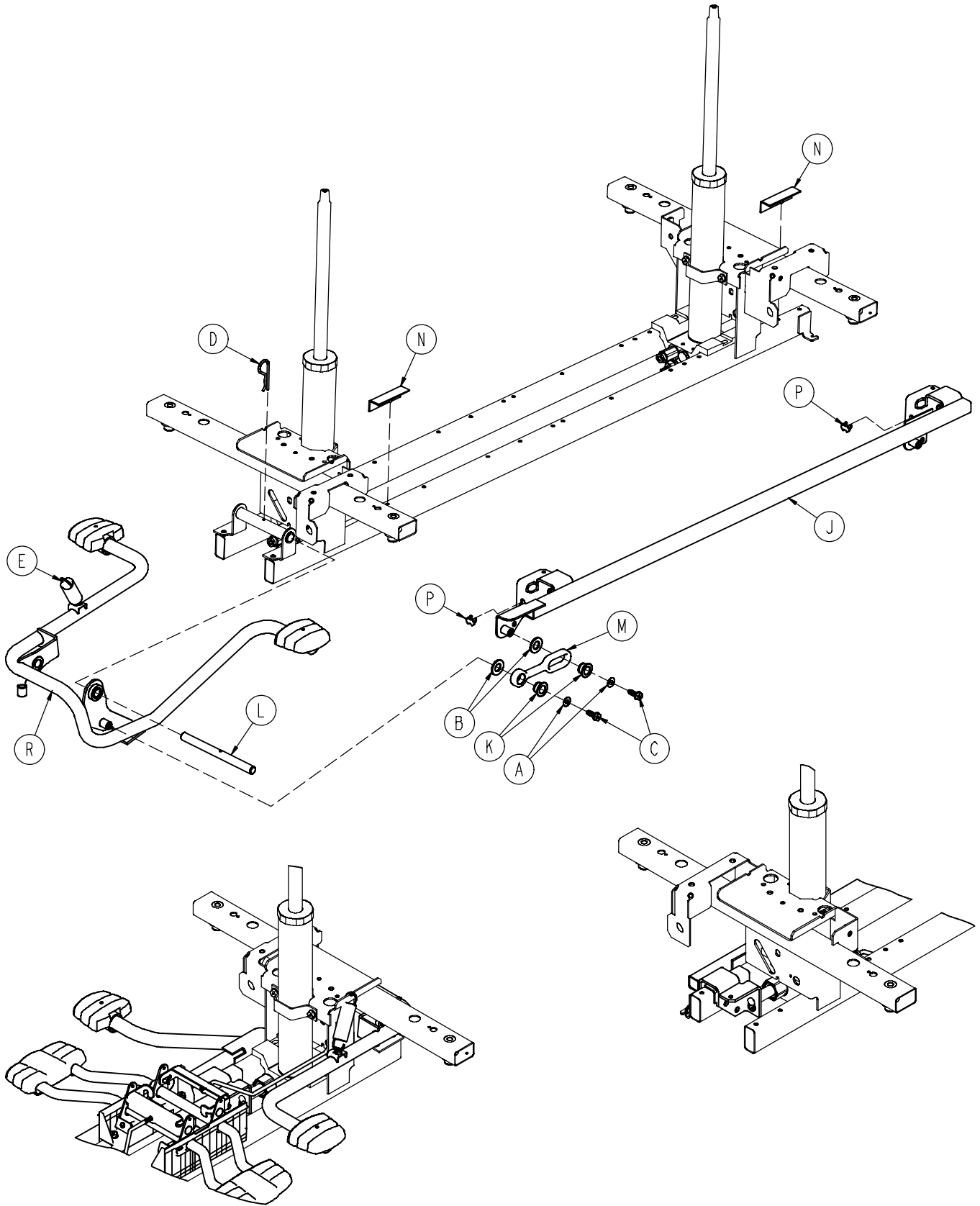

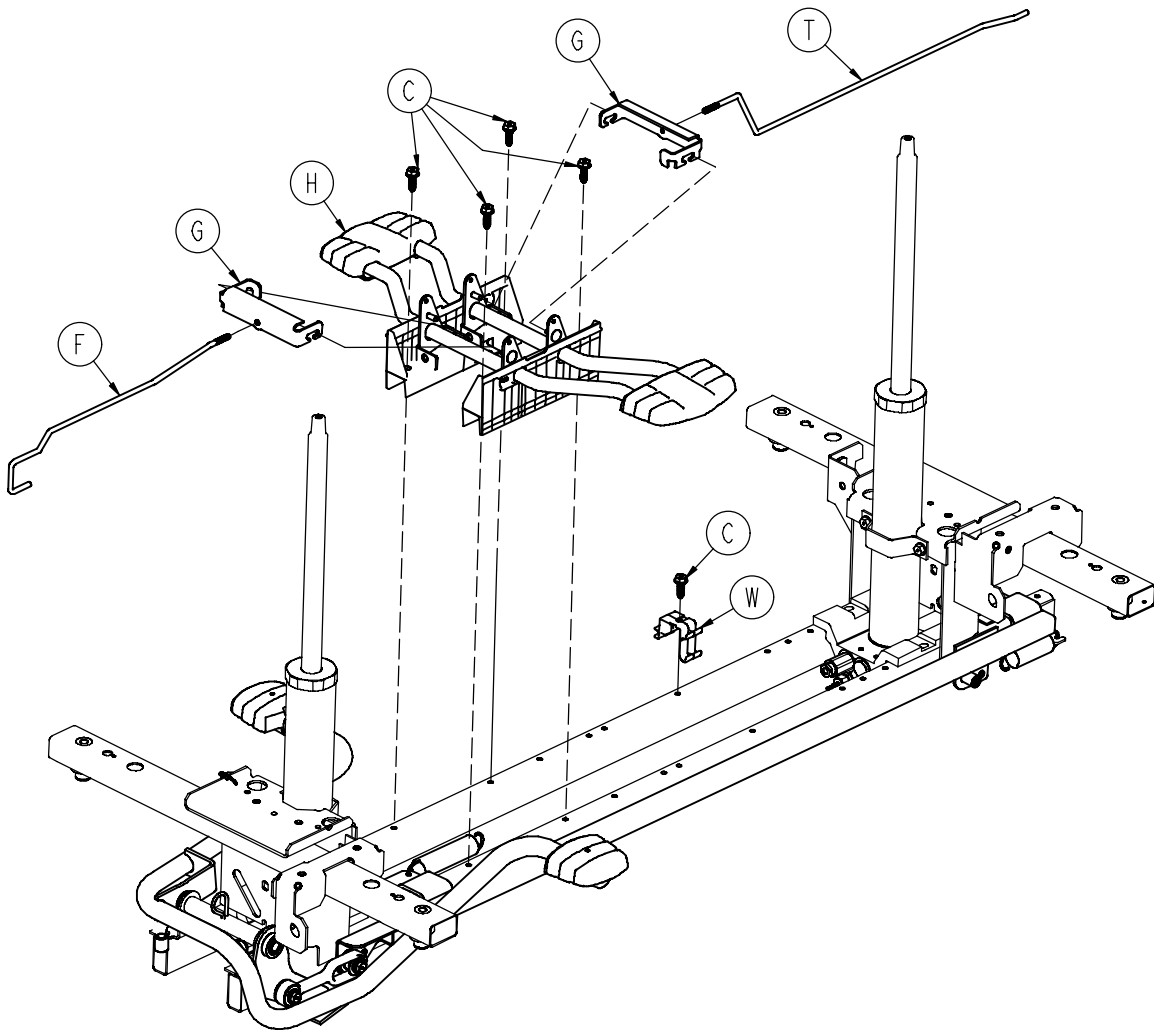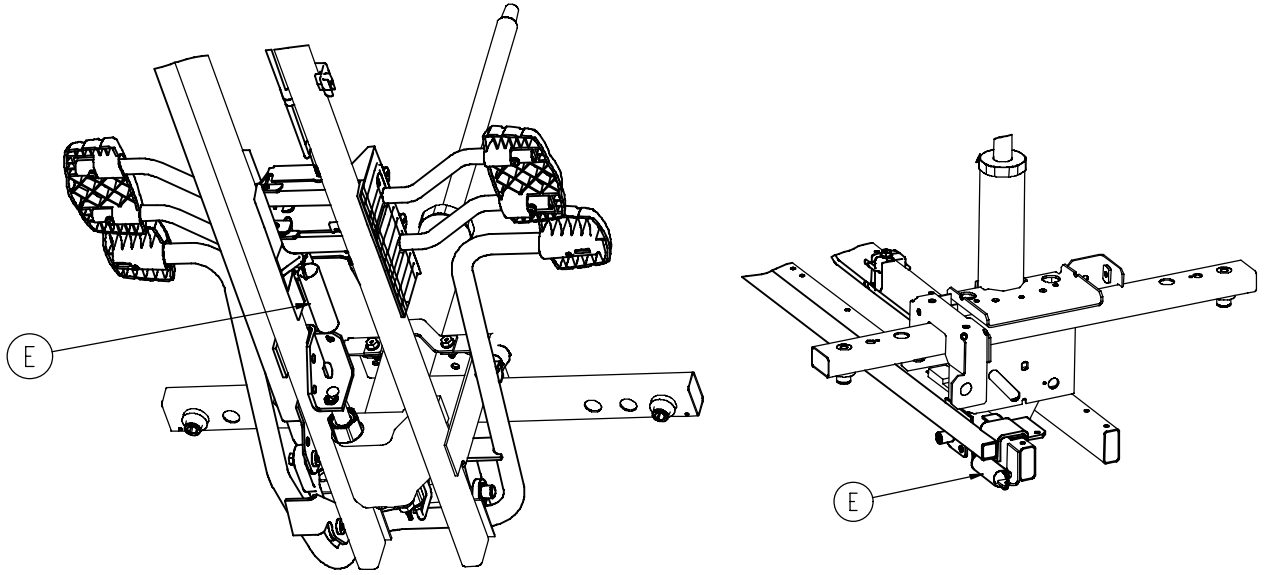


1040-005-500

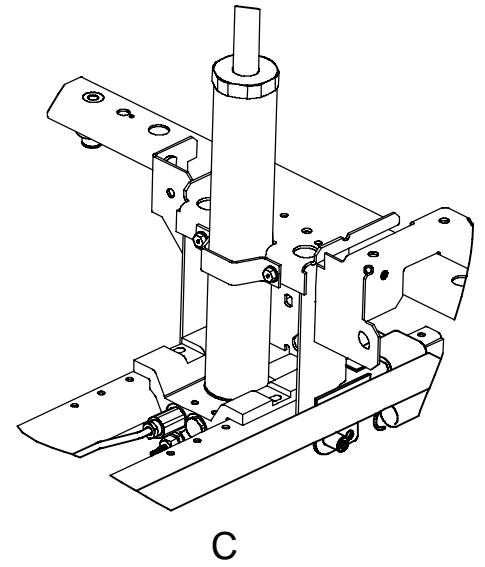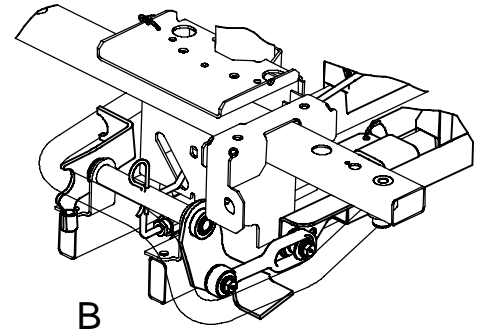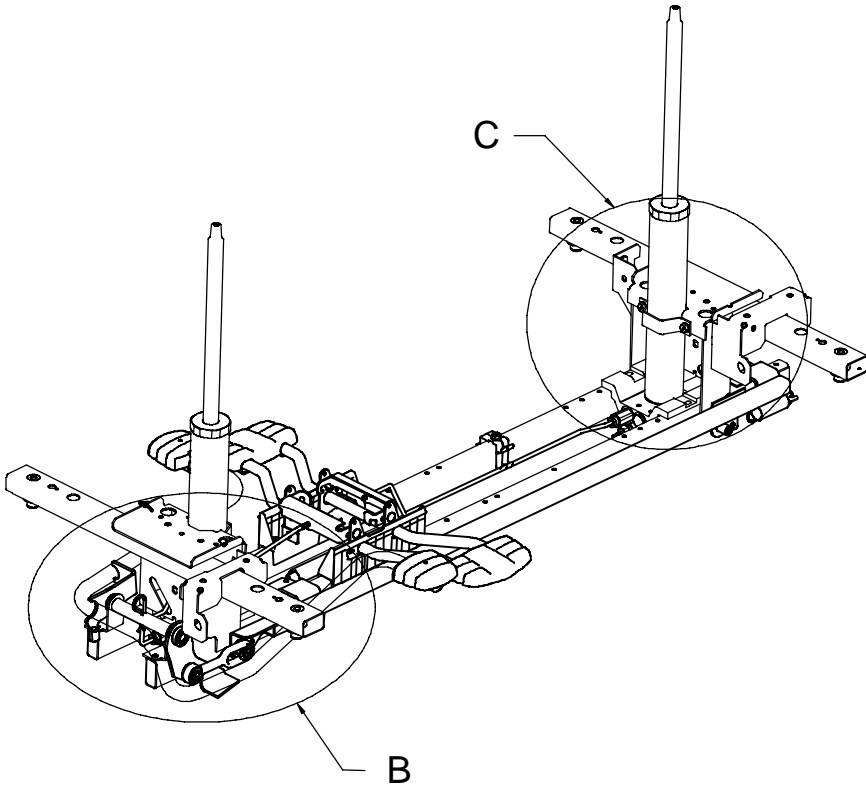


A	0011-023-000	2
B	0014-071-000	2
C	0023-288-000	7
D	0027-031-000	1
E	0038-497-000	3
F	0753-004-014	1
G	0753-004-132	2
H	0753-004-201	1
J	0753-005-037	1
K	0753-005-044	2
L	0753-005-074	1
M	0753-005-075	1
N	0753-005-088	2
P	0753-005-089	2
R	0753-005-285	1
T	1040-004-015	1
W	1040-010-215	1

0753-004-201

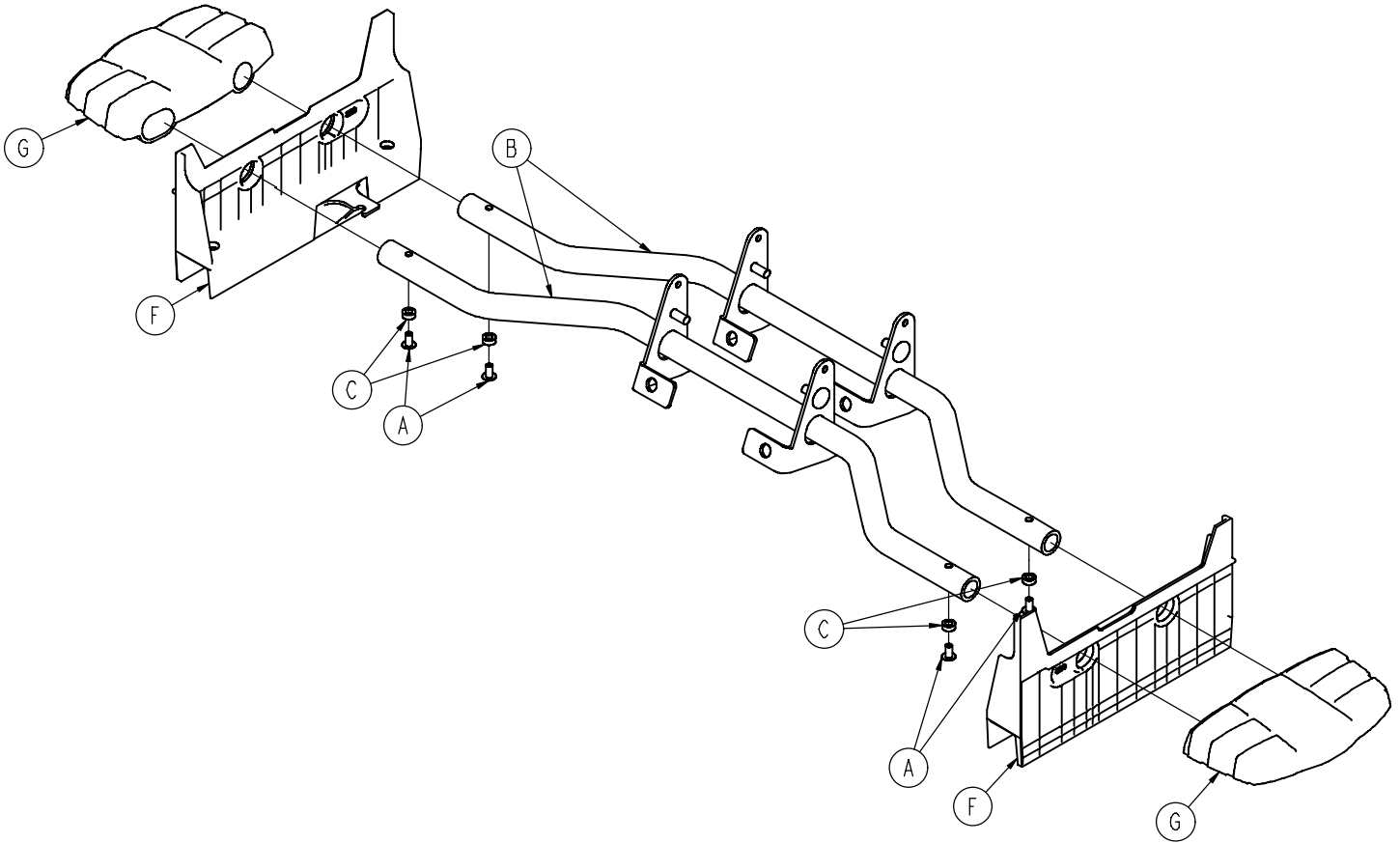


A	0025-079-000	4
B	0753-004-004	2
C	0753-004-029	4
F	0753-004-320	2
G	0785-005-127	2

0753-005-285

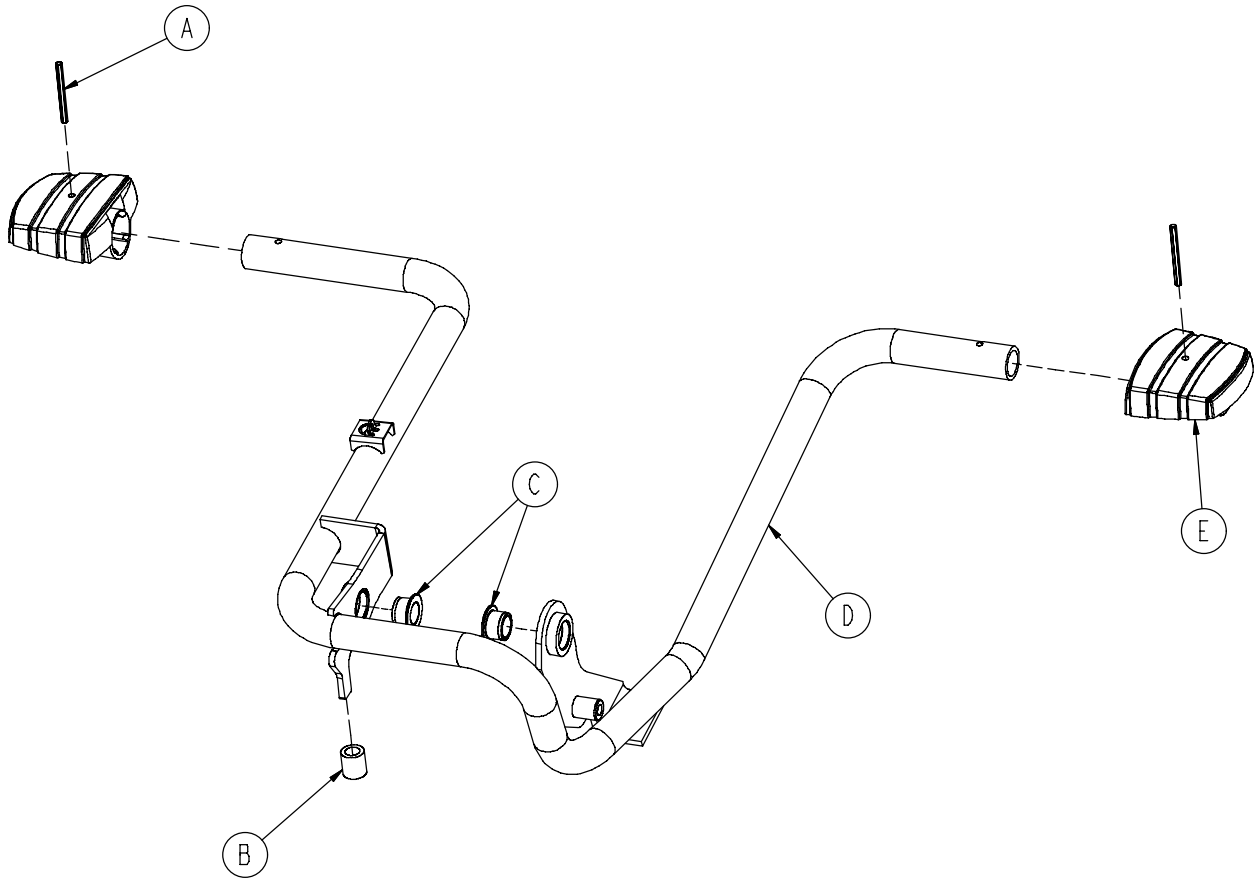


A	0026-343-000	2
B	0715-001-140	1
C	0753-005-044	2
D	0753-005-180	1
E	0785-005-126	2

1040-006-201

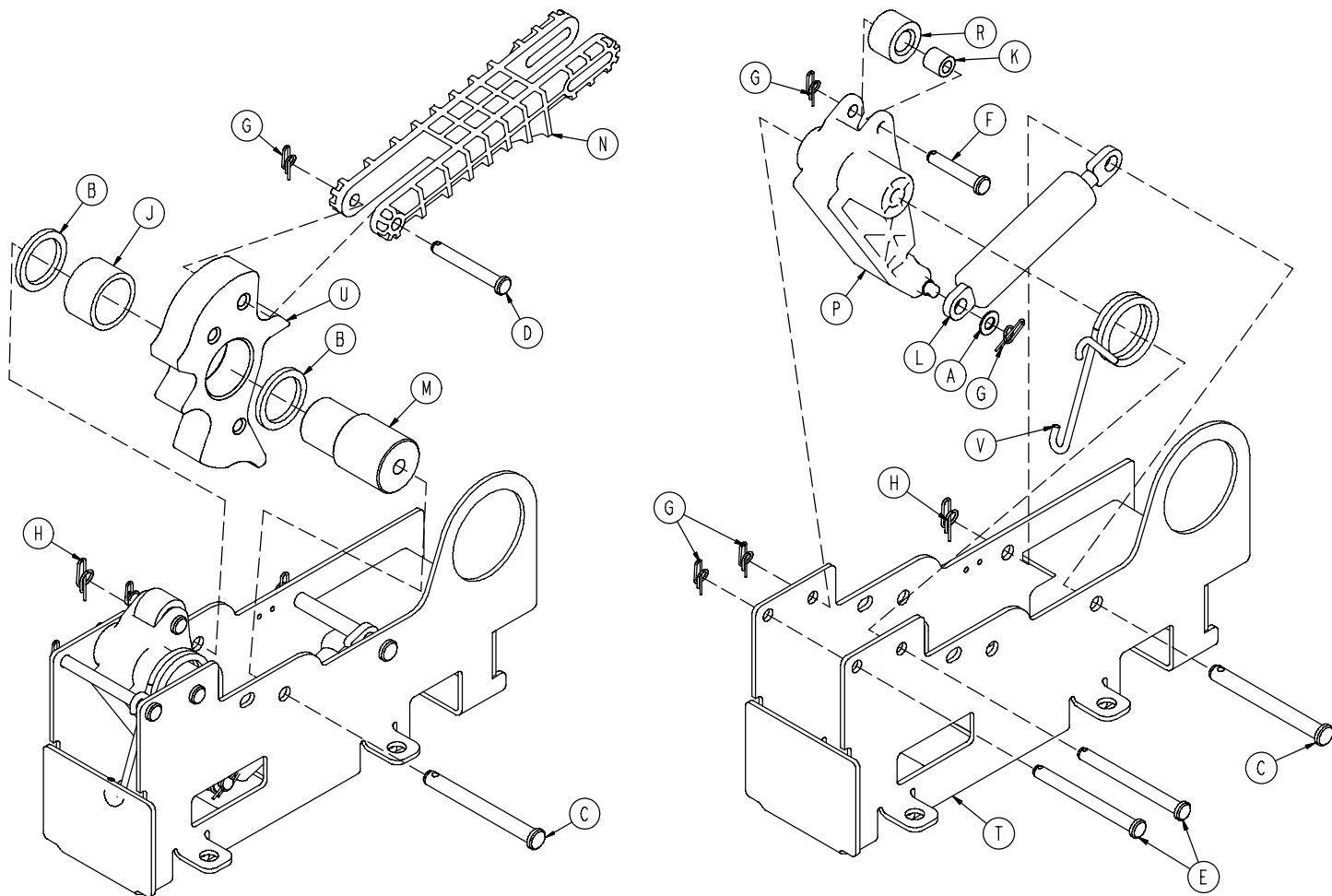




A	0014-004-000	1
B	0014-063-000	2
C	0023-262-000	2
D	0026-330-000	1
E	0026-341-000	2
F	0026-349-000	1
G	0027-020-000	5
H	0027-025-000	2
J	0081-400-000	1
K	0081-634-000	1
L	1040-006-209	1
M	1040-006-141	1
N	1040-006-144	1
P	1040-006-163	1
R	1040-006-171	1
T	1040-006-221	1
U	1040-006-313	1
V	1040-006-146	1

1040-015-201

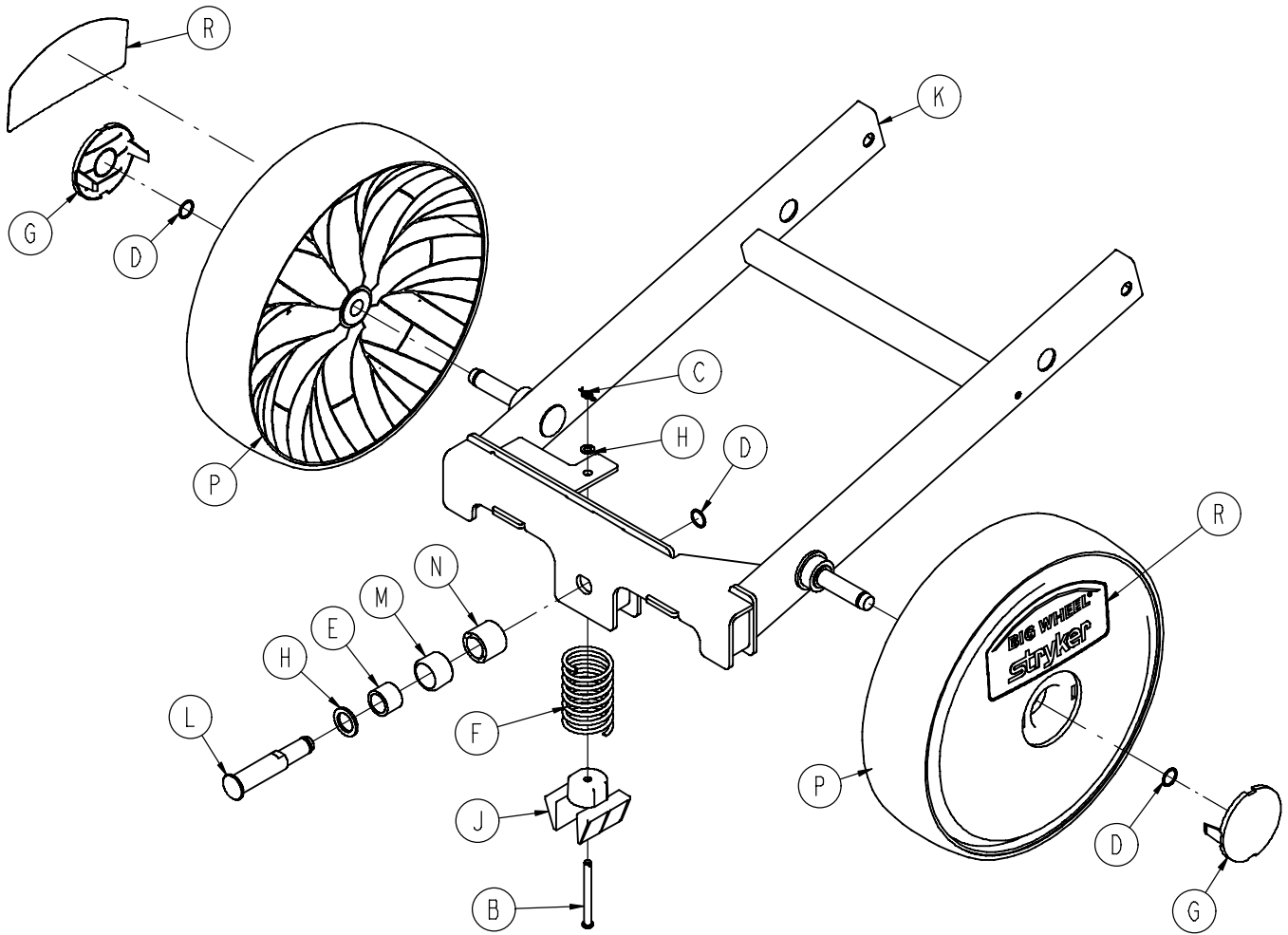

A	0014-004-000	1
B	0026-341-000	1
C	0027-020-000	1
D	0028-330-000	3
E	0081-011-000	1
F	1040-015-106	1
G	1040-015-073	2
H	1040-015-038	1
J	1040-015-053	1
K	1040-015-110	1
L	1040-020-122	1
M	1040-020-128	1
N	1040-020-129	1
P	1040-015-080	2
R	1040-015-078	2

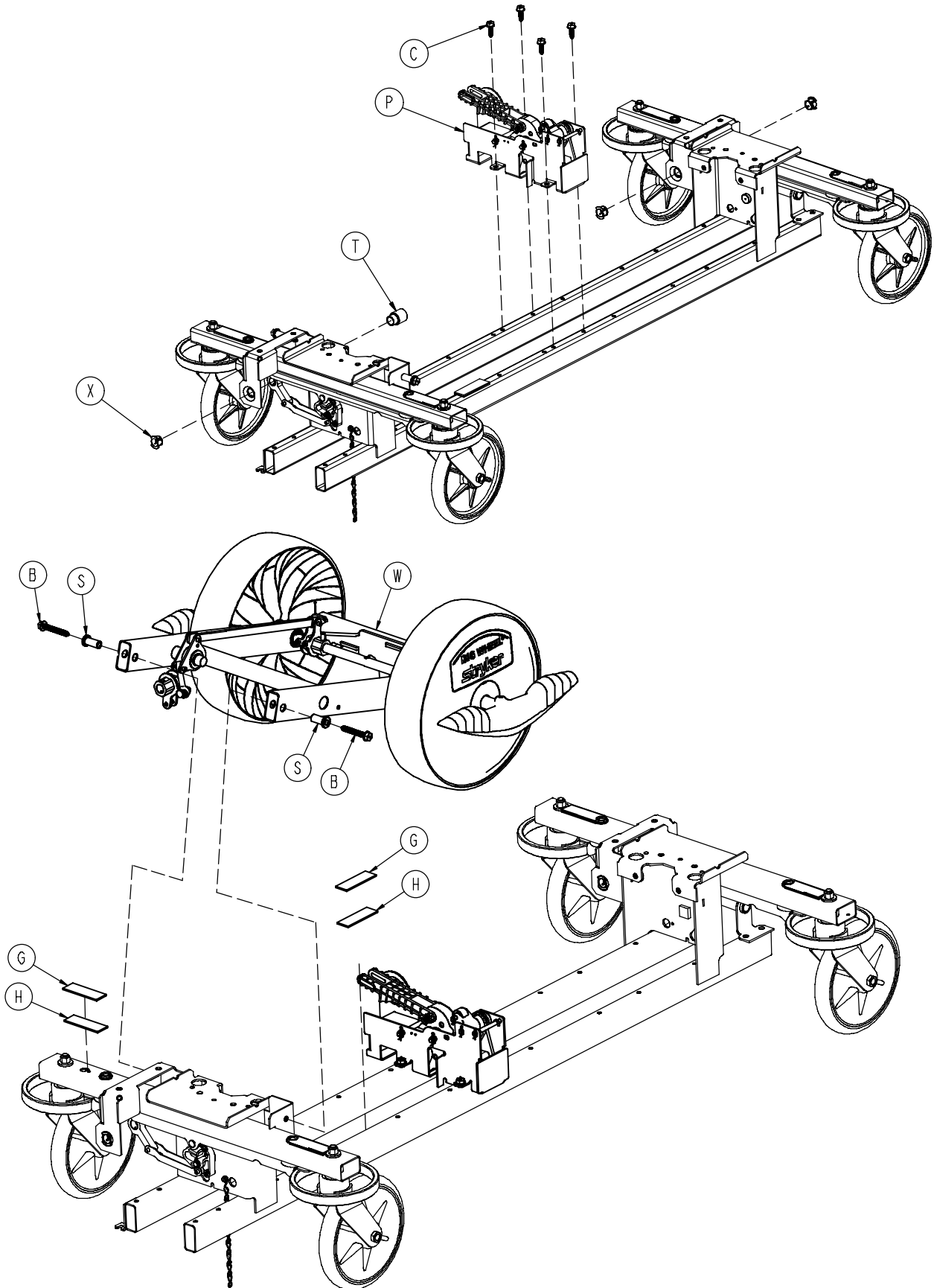


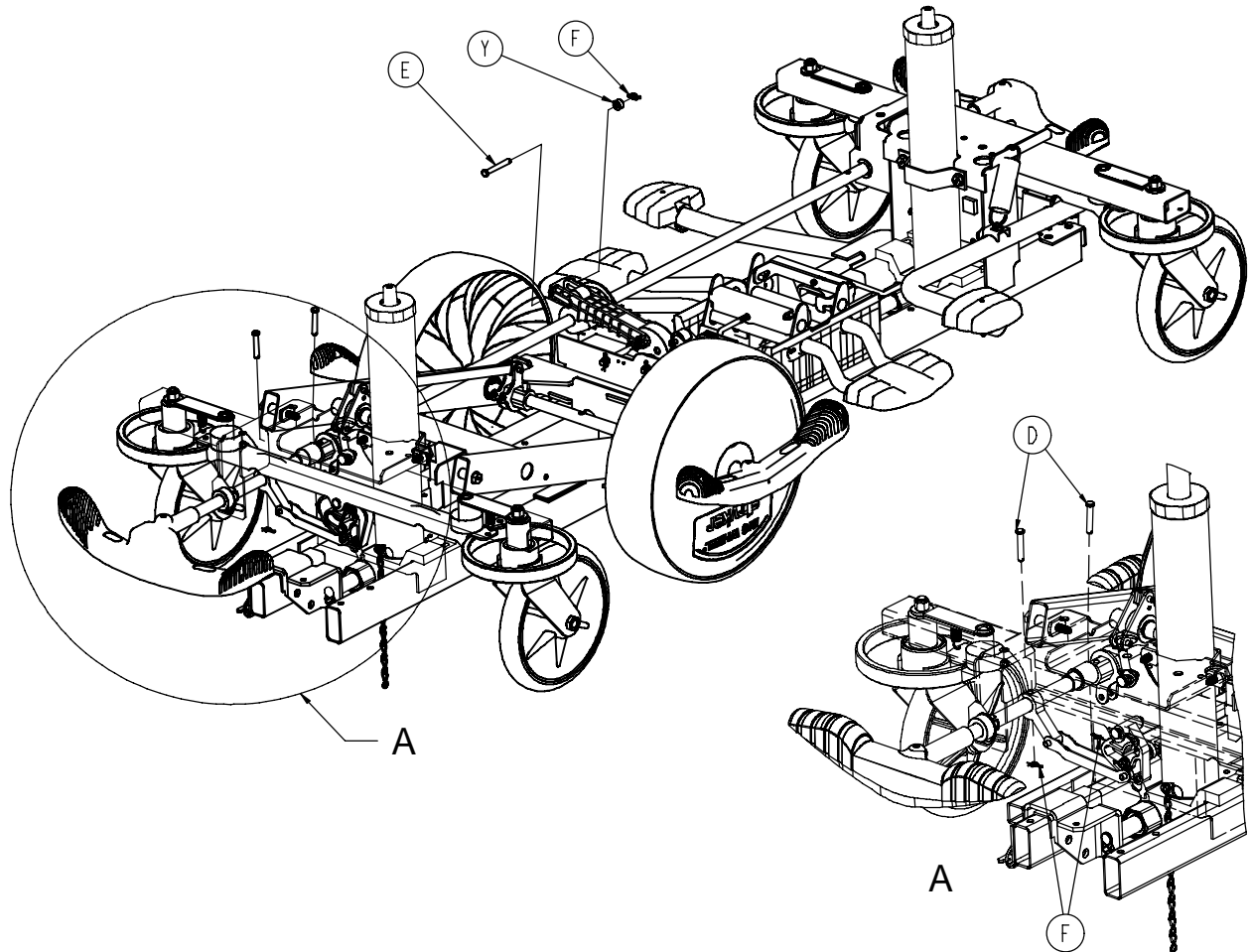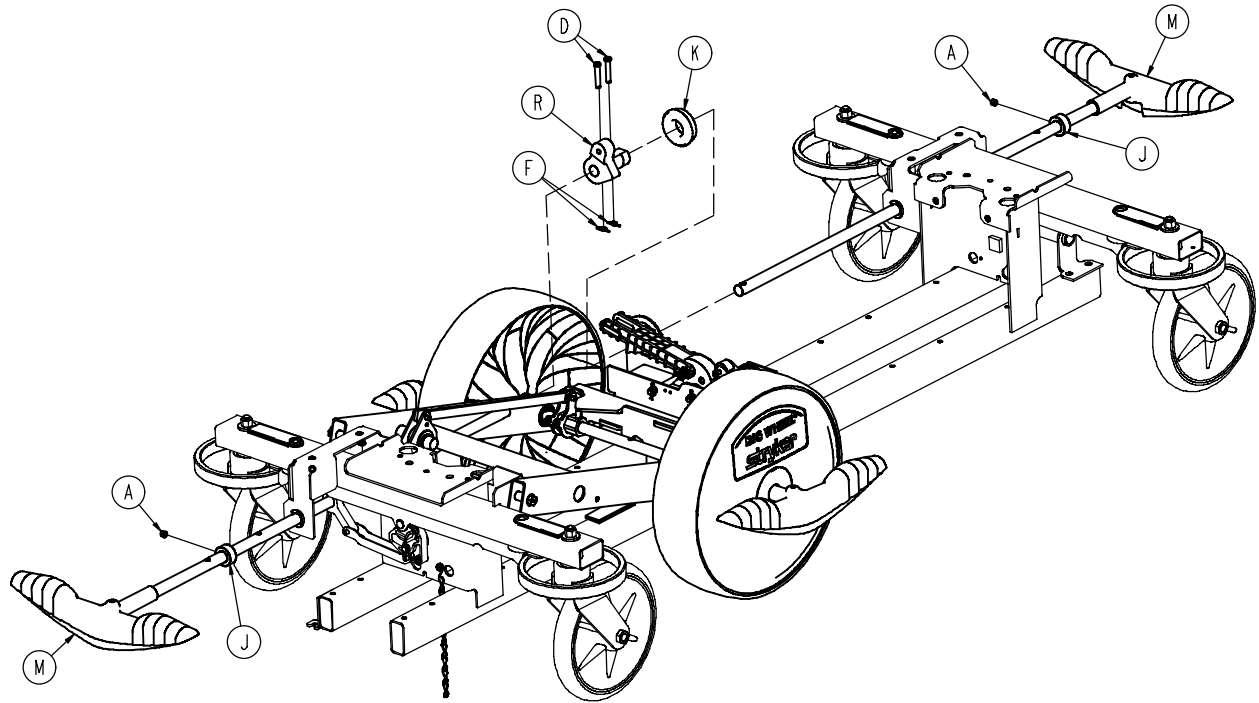


1040-025-205

A	0021-166-000	2
B	0023-150-000	2
C	0023-288-000	4
D	0026-273-000	5
E	0026-330-000	1
F	0027-020-000	6
G	0029-007-000	5
H	0029-009-000	5
J	0042-020-000	2
K	0081-330-000	1
L	1040-003-205	1
M	1040-003-291	2
N	1040-005-500	1
P	1040-006-201	1
R	1040-006-231	1
S	1040-015-007	2
T	1040-025-026	1
W	1040-025-201	1
X	1210-201-251	3
Y	0052-015-000	1


1040-025-010

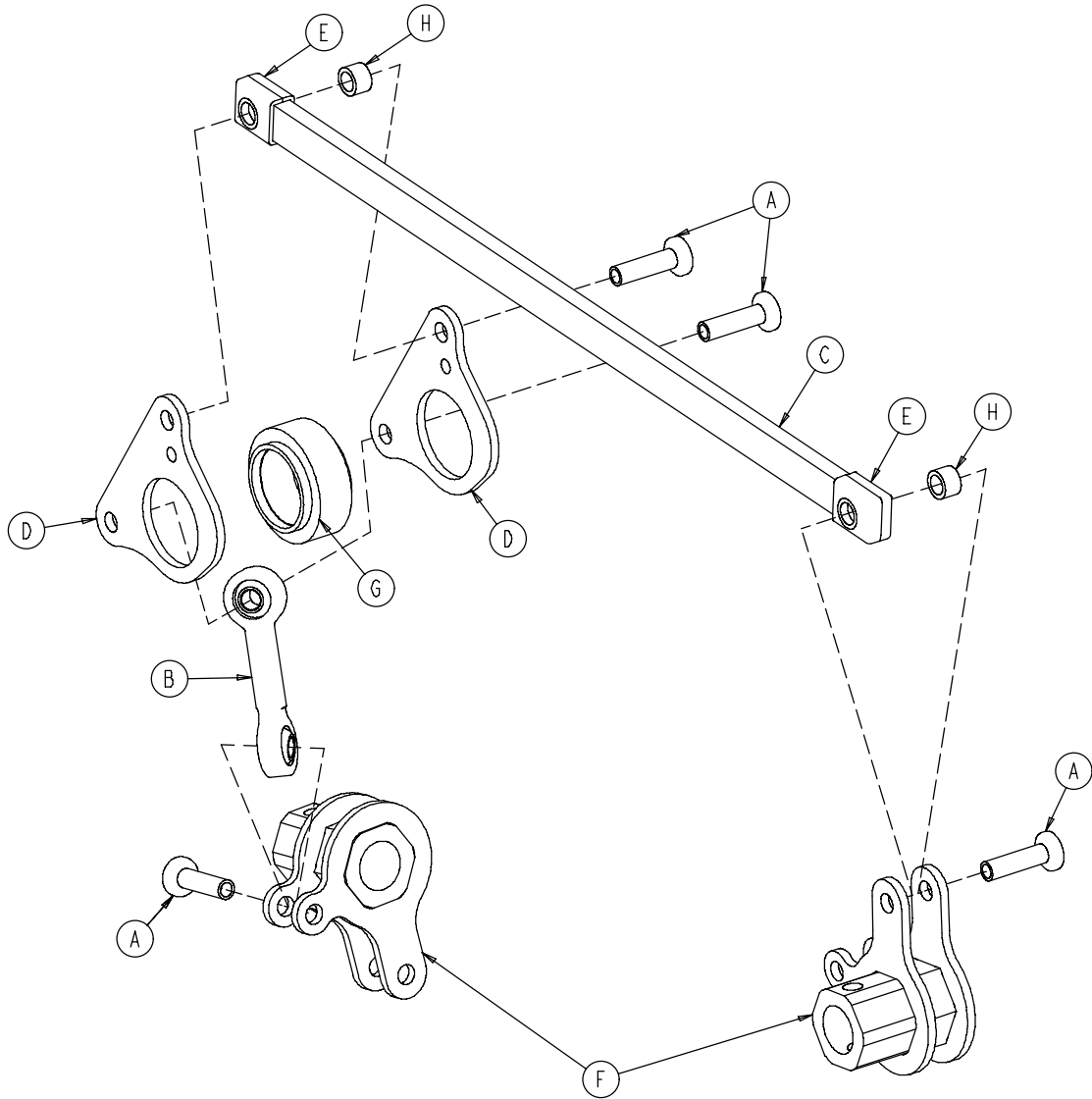


A	0753-003-098	4
B	0753-003-117	1
C	1040-025-011	1
D	1040-025-013	2
E	1040-025-016	2
F	1040-025-020	2
G	1040-025-018	1
H	1040-025-027	2

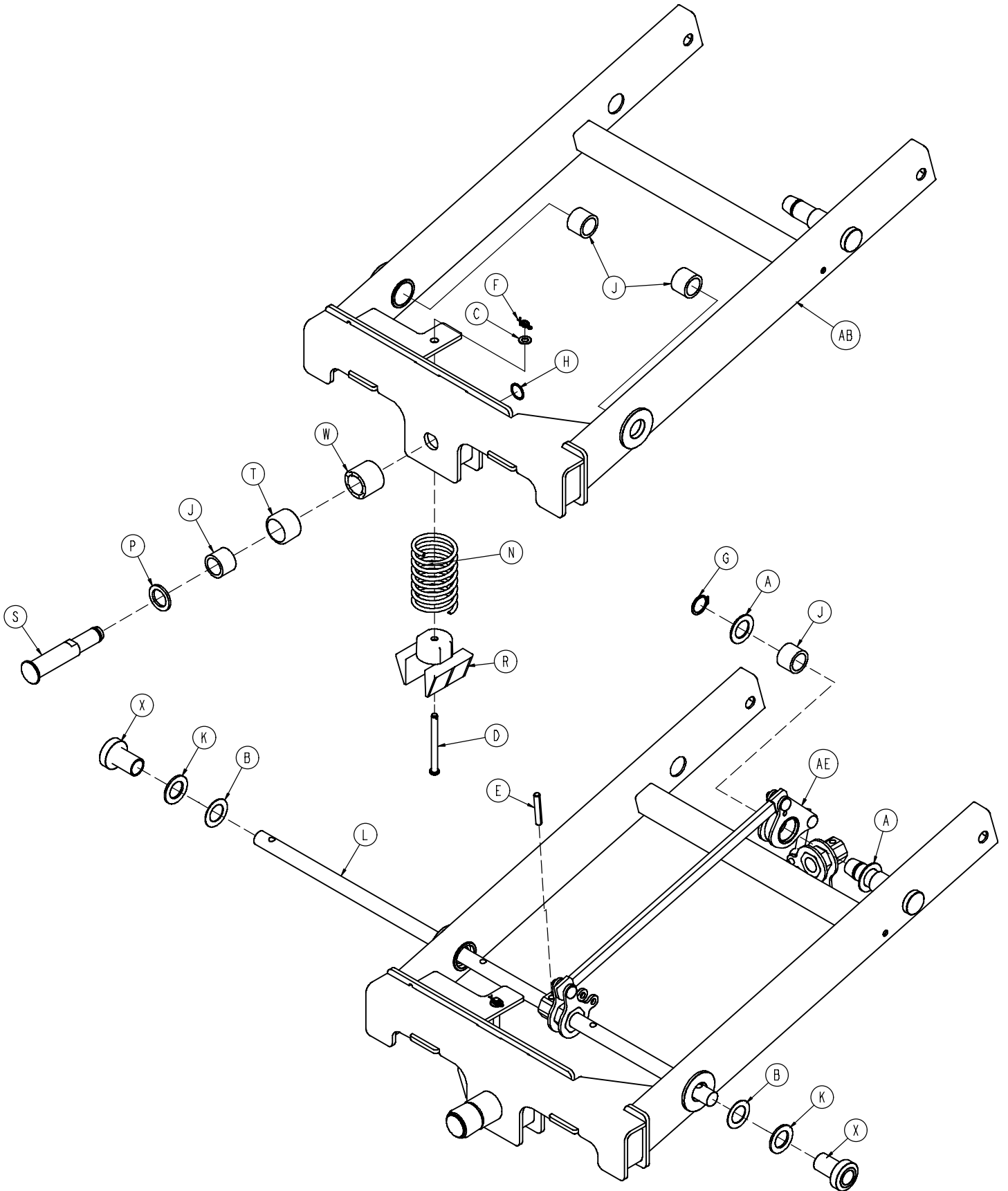


1040-025-201

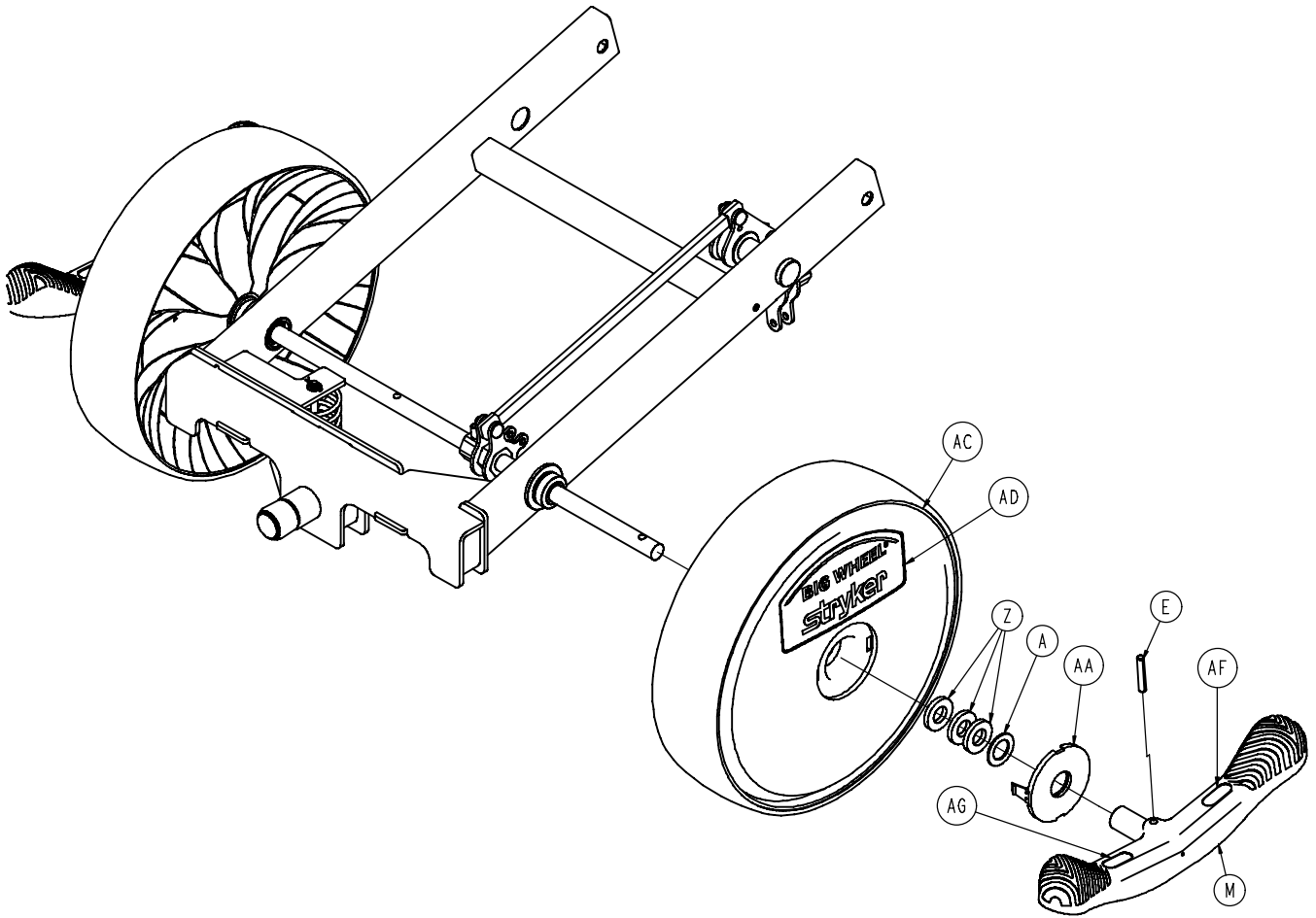

A	0011-148-000	4	P	1040-015-038	1
B	0011-525-000	2	R	1040-015-053	1
C	0014-004-000	1	S	1040-020-122	1
D	0026-341-000	1	T	1040-020-128	1
E	0026-342-000	3	W	1040-020-129	1
F	0027-020-000	1	X	1040-025-014	2
G	0028-166-000	1	Z	1040-025-017	6
H	0028-330-000	1	AA	1040-015-074	2
J	0081-011-000	4	AB	1040-025-110	1
K	0081-175-000	2	AC	1040-015-080	2
L	0753-003-219	1	AD	1040-015-078	2
M	1040-005-155	2	AE	1040-025-010	1
N	1040-015-106	1	AF	1210-201-335	2
			AG	1210-201-336	2

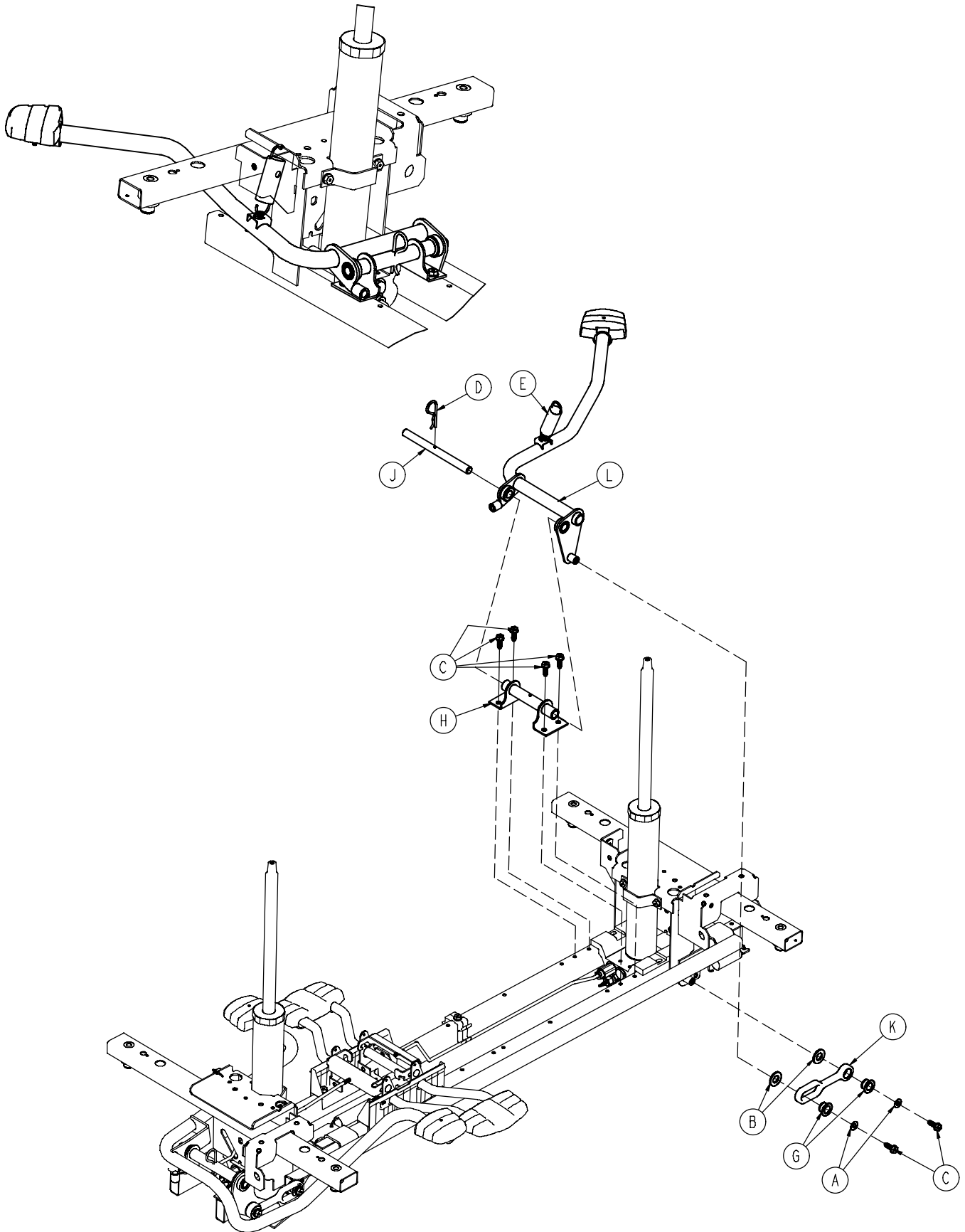


0753-005-610

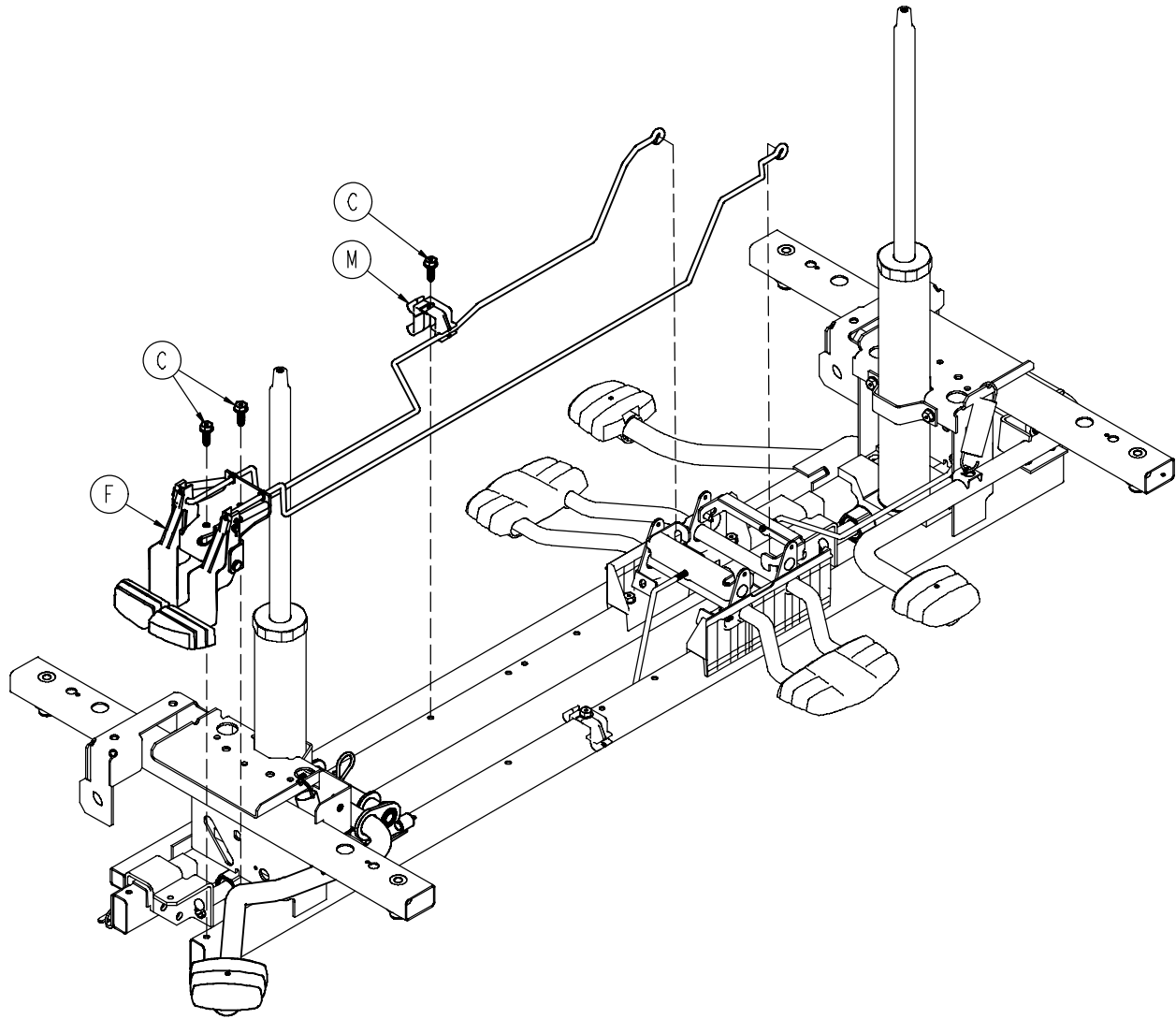


A	0011-023-000	2
B	0014-071-000	2
C	0023-288-000	9
D	0027-031-000	1
E	0038-497-000	1
F	0753-004-423	1
G	0753-005-044	2
H	0753-005-063	1
J	0753-005-074	1
K	0753-005-075	1
L	0753-005-273	1
M	1040-010-215	1

0753-004-423

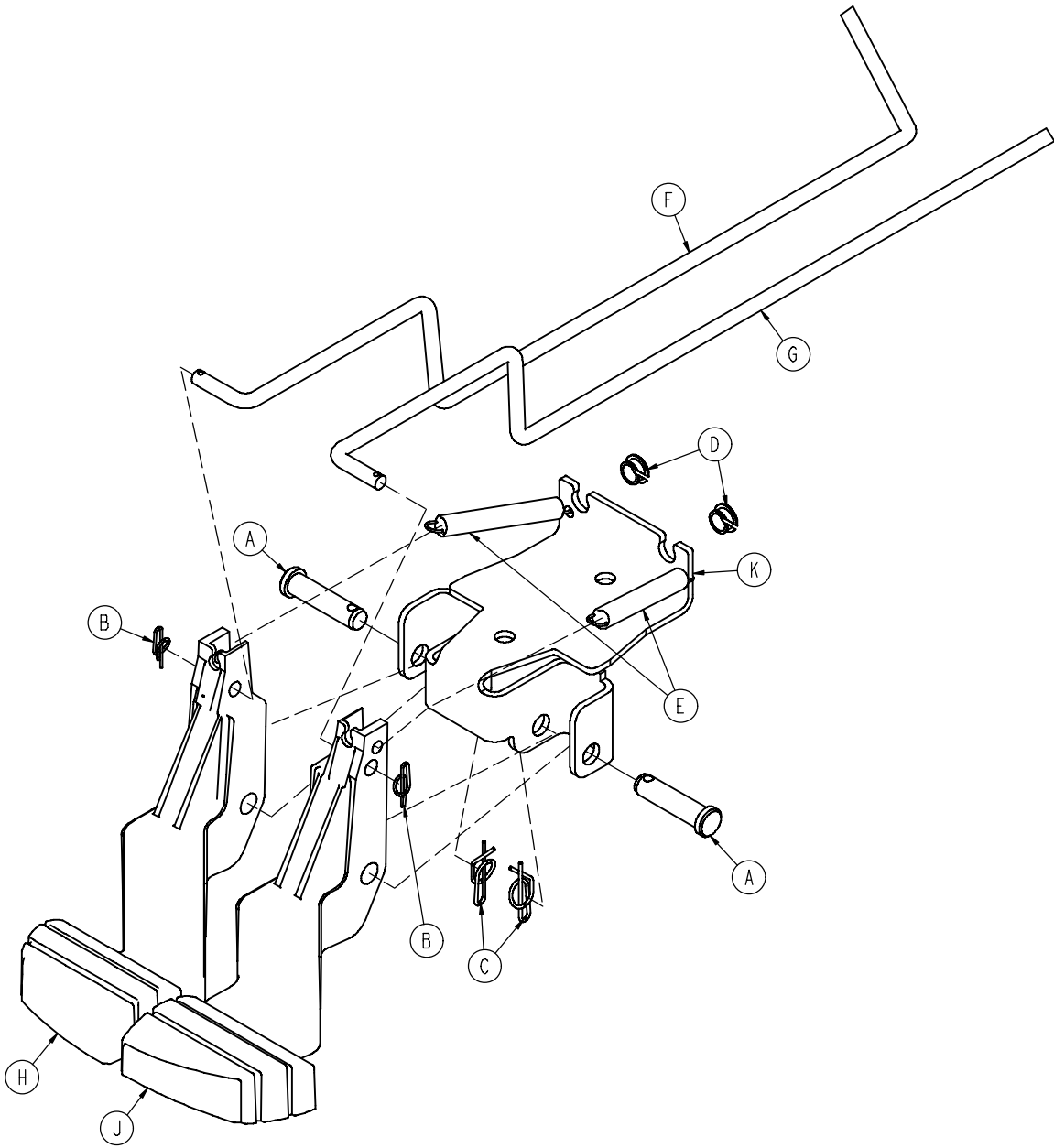


A	0026-132-000	2
B	0027-020-000	2
C	0027-022-000	2
D	0052-822-000	2
E	0753-004-028	2
F	0753-004-038	1
G	0753-004-039	1
H	0753-004-144	1
J	0753-004-145	1
K	0753-004-422	1

0753-005-273

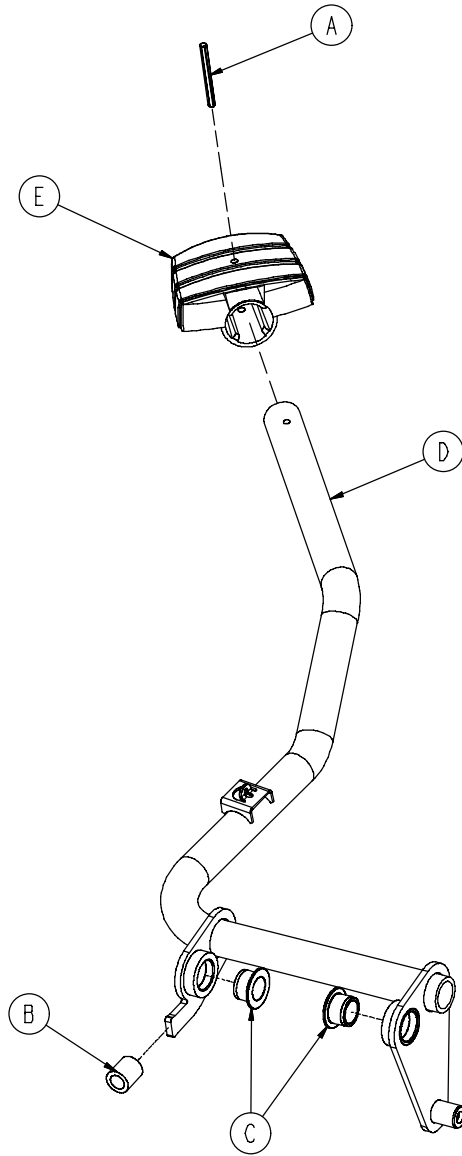


A	0026-343-000	1
B	0715-001-140	1
C	0753-005-044	2
D	0753-005-171	1
E	0785-005-126	1

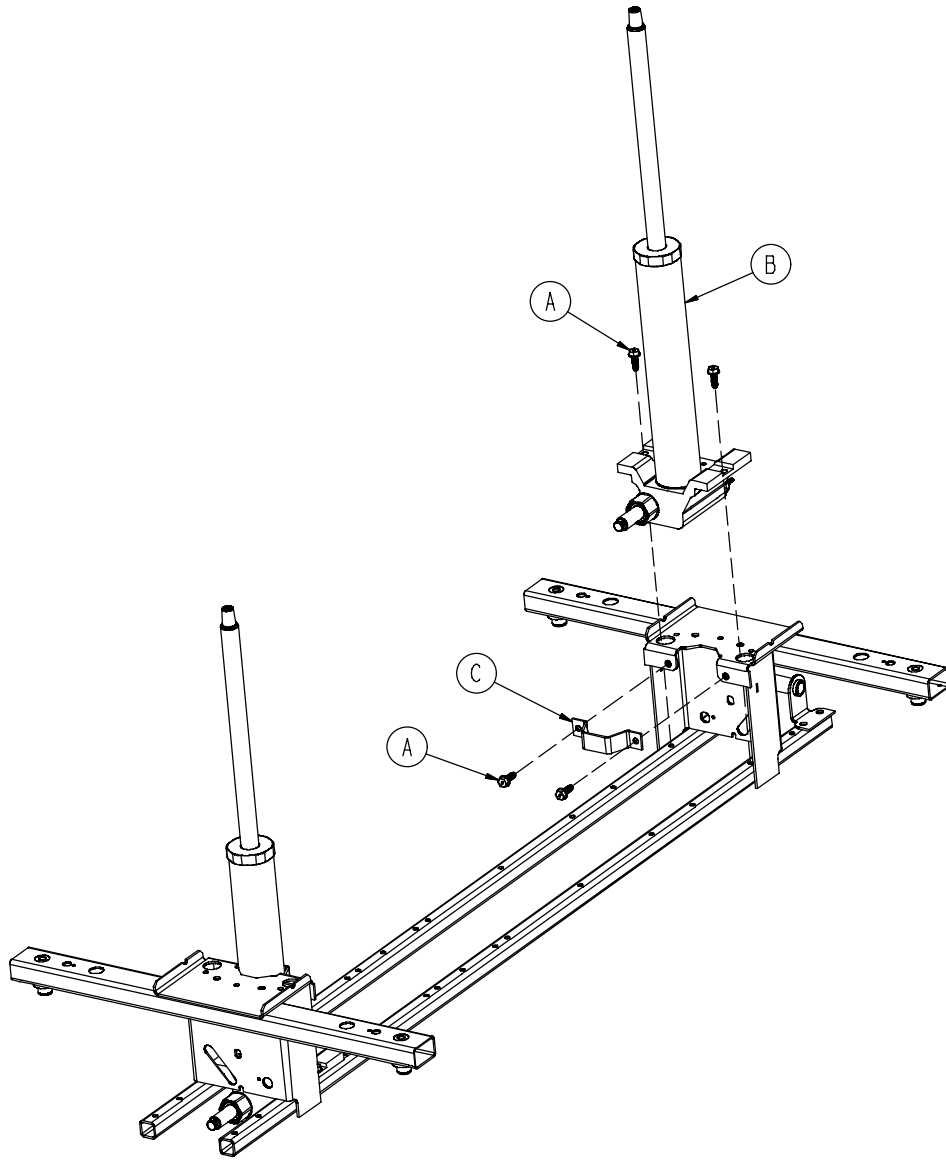


A	0023-288-000	8
B	0753-002-101	2
C	0753-010-007	2

0753-002-101

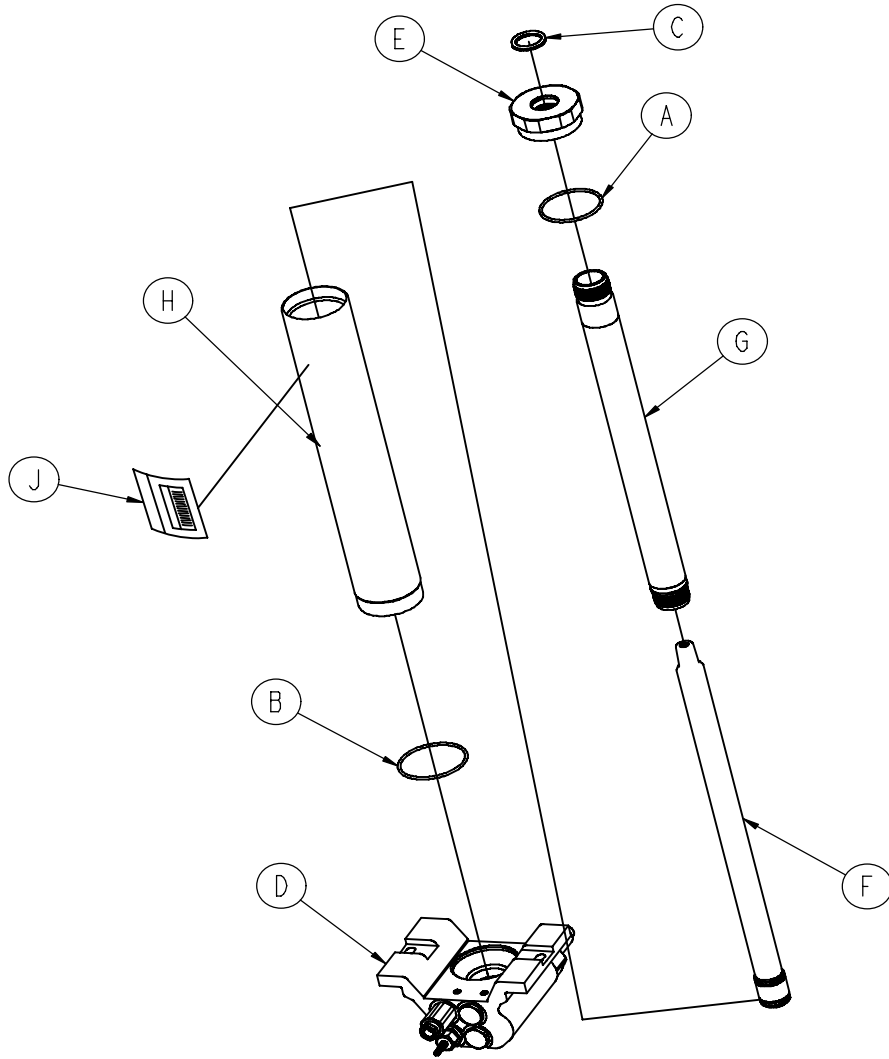






A	0045-273-000	1
B	0045-274-000	1
C	0048-190-000	1
D	0753-002-002	1
E	0753-002-017	1
F	0753-002-104	1
G	0753-002-106	1
H	0753-102-129	1
J	0753-202-100	1

0753-002-002

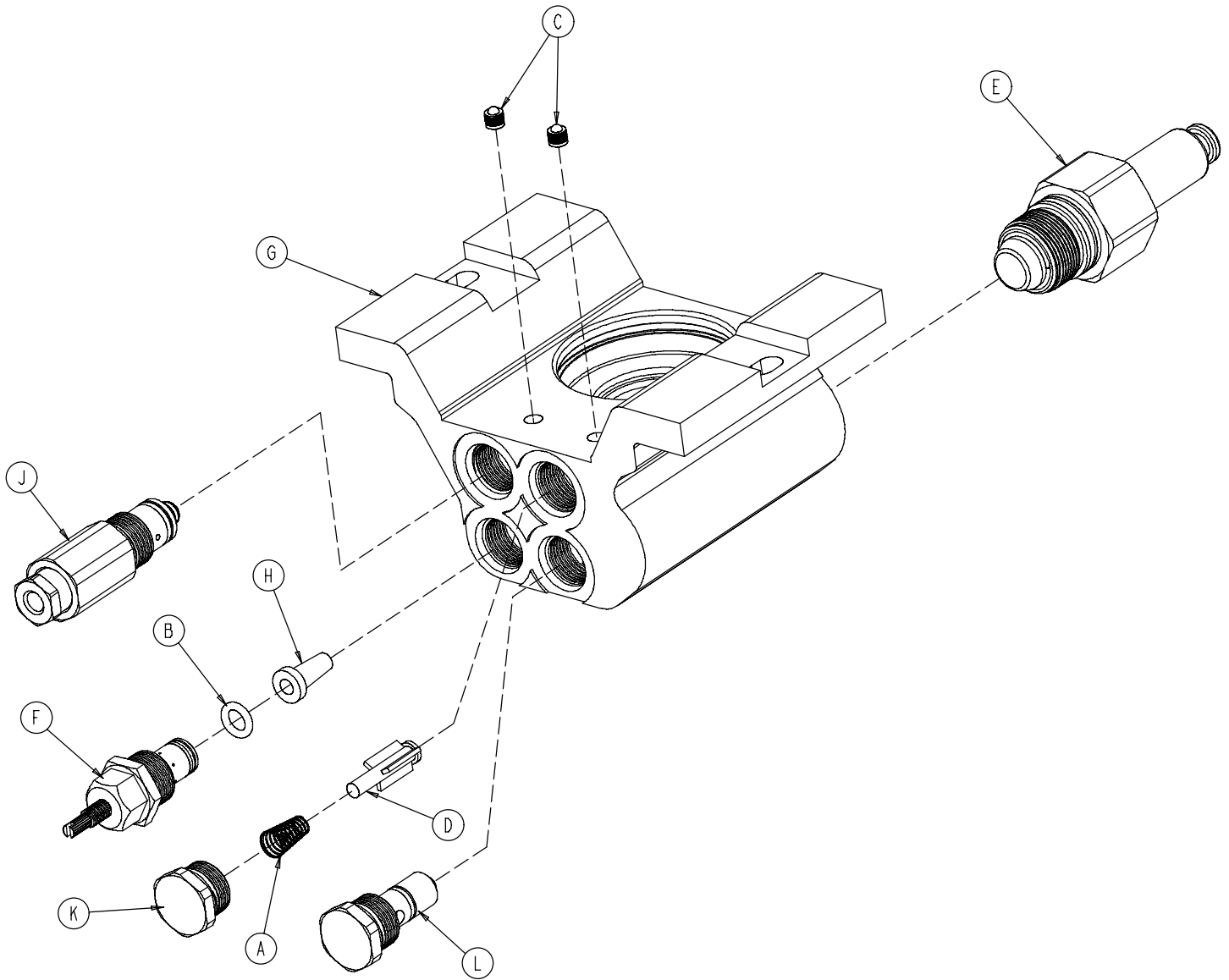

A	0390-002-134	1
B	0046-004-000	1
C	0048-021-000	2
D	0715-001-341	1
E	0753-002-003	1
F	0753-002-058	1
G	0753-002-010	1
H	0753-002-019	1
J	0753-002-065	1
K	0753-002-051	1
L	0753-002-052	1

1015-026-100

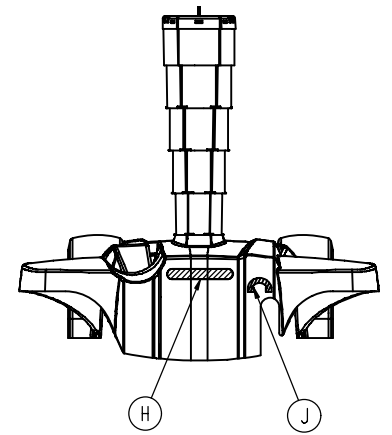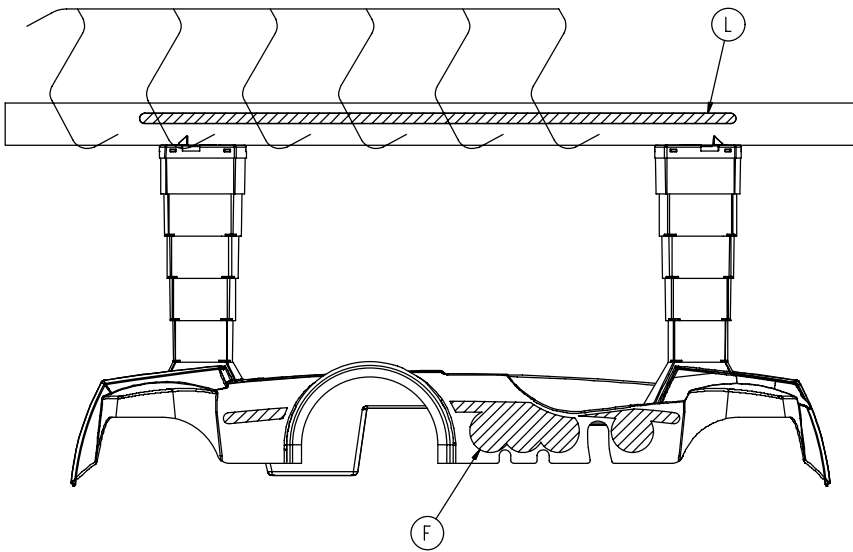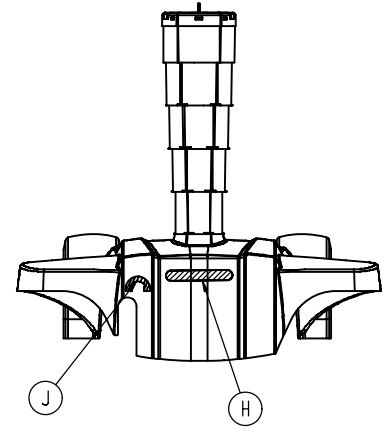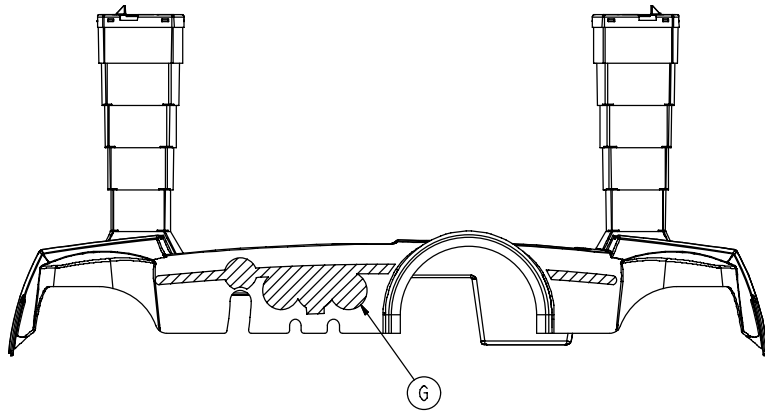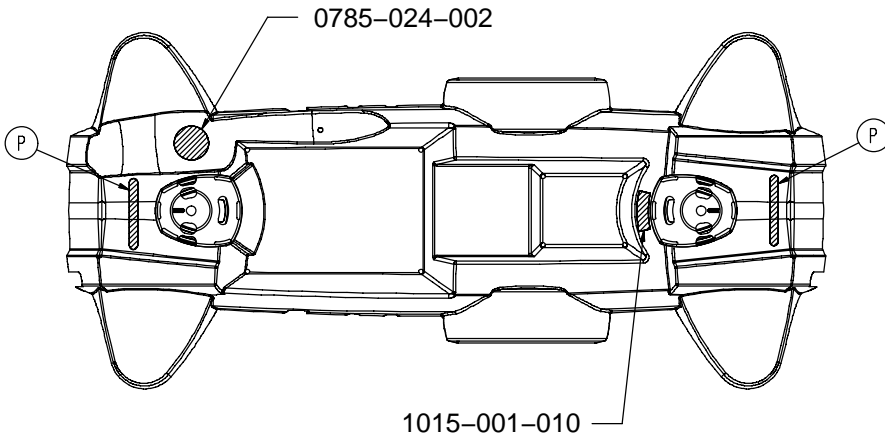

A
B






0785-001-815
1040-210-035

2
1

1015-026-100

F	G	H	J	K	L
0785-026-101	0785-026-201	0785-024-801	0785-025-600	0785-025-700	0785-022-101
0785-026-102	0785-026-202	0785-024-802	0785-025-600	0785-025-700	0785-022-102
0785-026-103	0785-026-203	0785-024-803	0785-025-600	0785-025-700	0785-022-103
0785-026-104	0785-026-204	0785-024-804	0785-025-600	0785-025-700	0785-022-104
0785-026-105	0785-026-205	0785-024-805	0785-025-600	0785-025-700	0785-022-105
0785-026-106	0785-026-206	0785-024-806	0785-025-600	0785-025-700	0785-022-106

1015-026-200

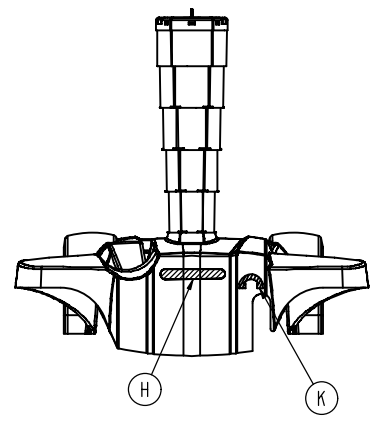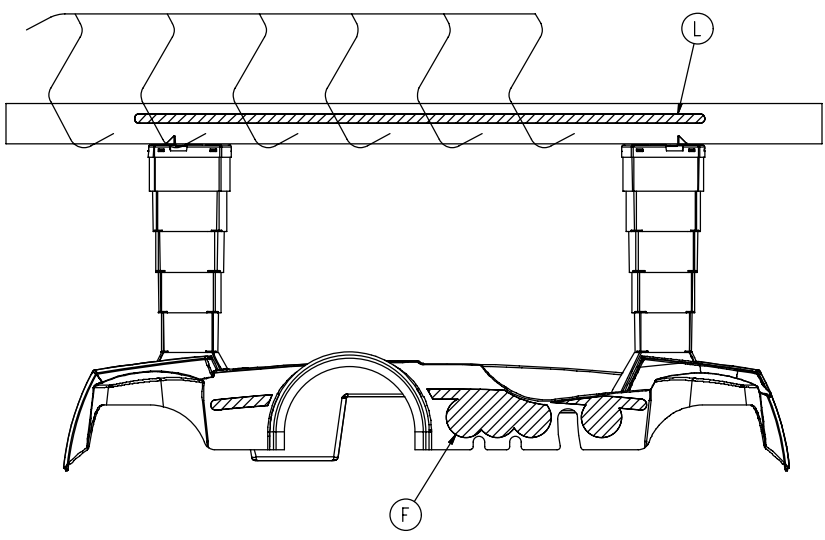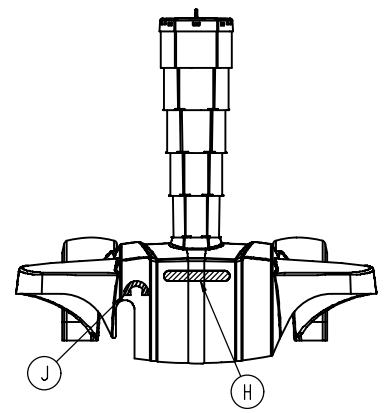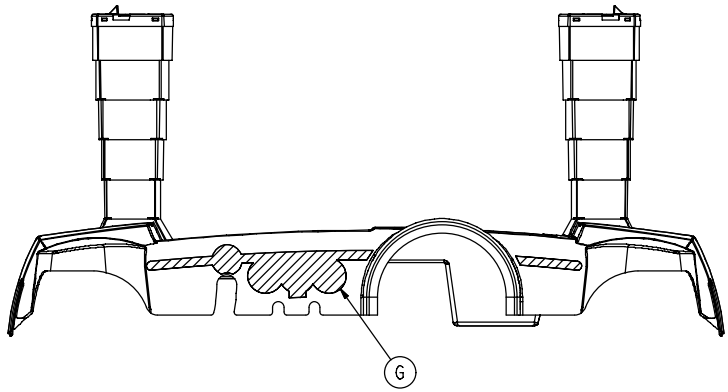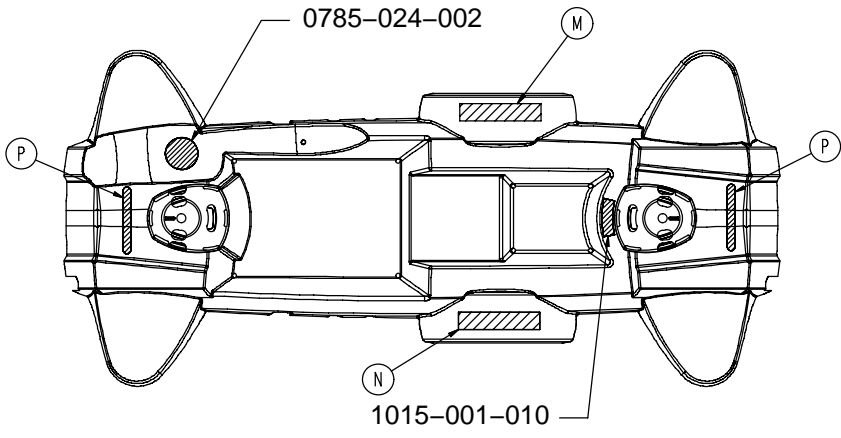

A 0785-001-815
 B 1040-210-035






2
 1

1015-026-200

F	G	H	J	K
0785-026-101	0785-026-201	0785-024-801	0785-025-600	0785-025-700
0785-026-102	0785-026-202	0785-024-802	0785-025-600	0785-025-700
0785-026-103	0785-026-203	0785-024-803	0785-025-600	0785-025-700
0785-026-104	0785-026-204	0785-024-804	0785-025-600	0785-025-700
0785-026-105	0785-026-205	0785-024-805	0785-025-600	0785-025-700
0785-026-106	0785-026-206	0785-024-806	0785-025-600	0785-025-700

L	M	N
0785-022-101	0785-026-300	0785-026-400
0785-022-102	0785-026-300	0785-026-400
0785-022-103	0785-026-300	0785-026-400
0785-022-104	0785-026-300	0785-026-400
0785-022-105	0785-026-300	0785-026-400
0785-022-106	0785-026-300	0785-026-400

1015-026-300

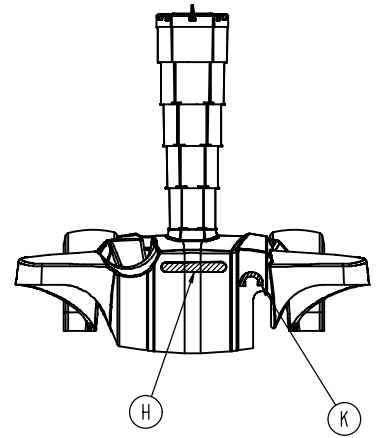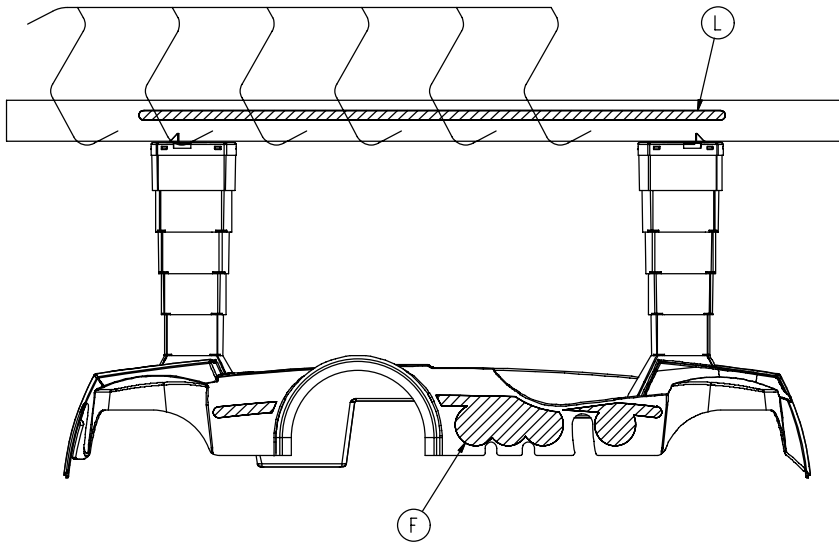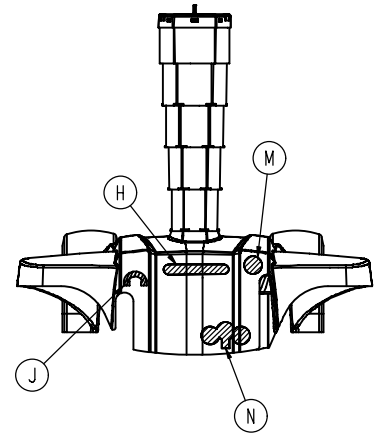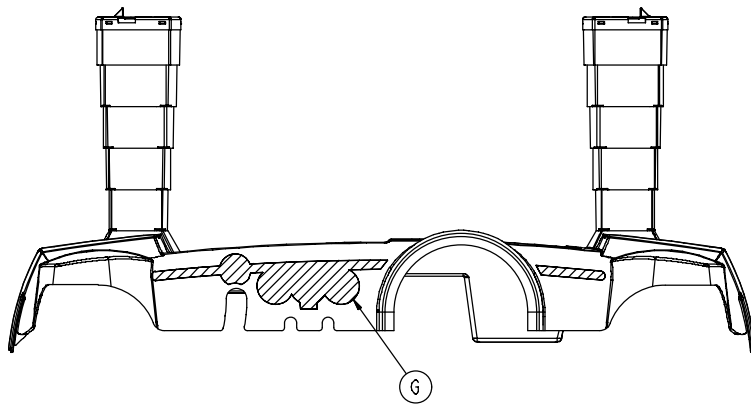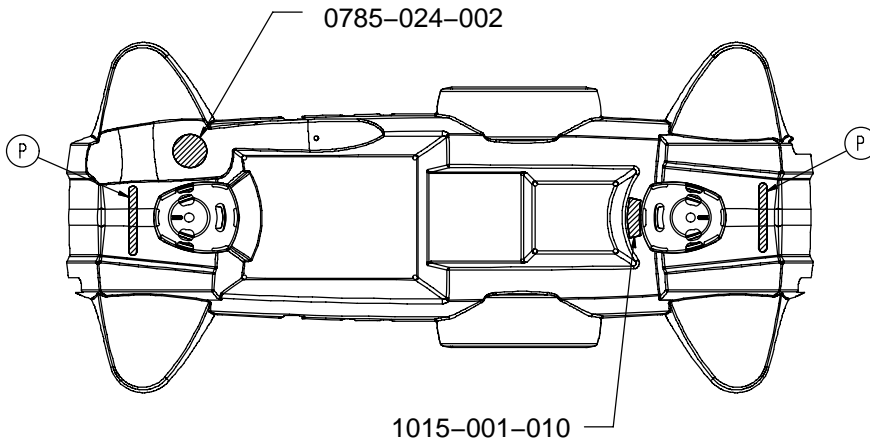

A
B

0785-001-815
1040-210-036






2
1

1015-026-300

F	G	H	J	K
0785-026-101	0785-026-201	0785-024-801	0785-025-600	0785-025-700
0785-026-102	0785-026-202	0785-024-802	0785-025-600	0785-025-700
0785-026-103	0785-026-203	0785-024-803	0785-025-600	0785-025-700
0785-026-104	0785-026-204	0785-024-804	0785-025-600	0785-025-700
0785-026-105	0785-026-205	0785-024-805	0785-025-600	0785-025-700
0785-026-106	0785-026-206	0785-024-806	0785-025-600	0785-025-700

L	M	N
0785-022-101	0785-024-501	0785-024-901
0785-022-102	0785-024-502	0785-024-902
0785-022-103	0785-024-503	0785-024-903
0785-022-104	0785-024-504	0785-024-904
0785-022-105	0785-024-505	0785-024-905
0785-022-106	0785-024-506	0785-024-906

1015-026-400

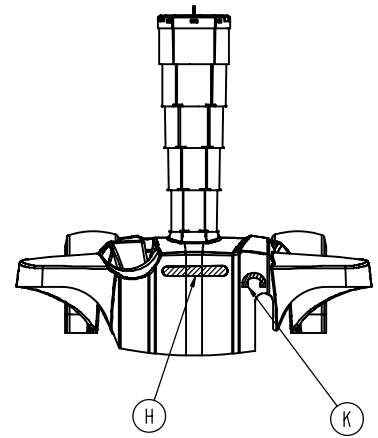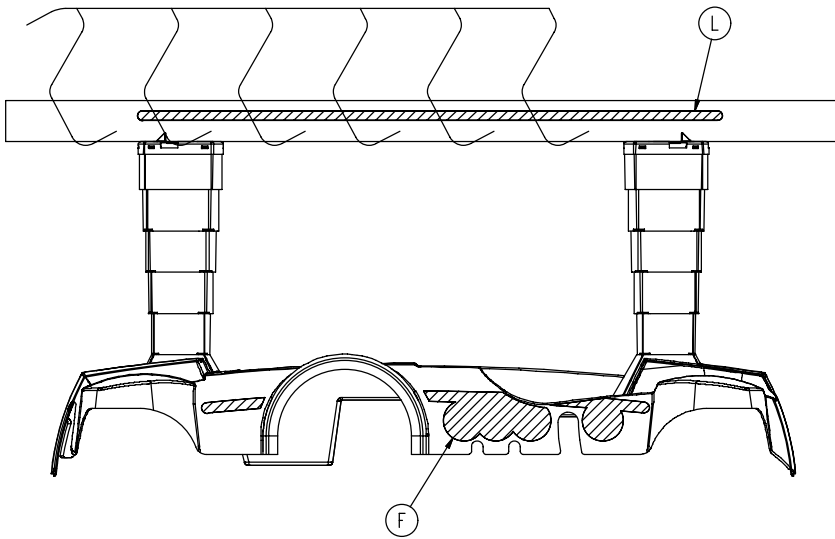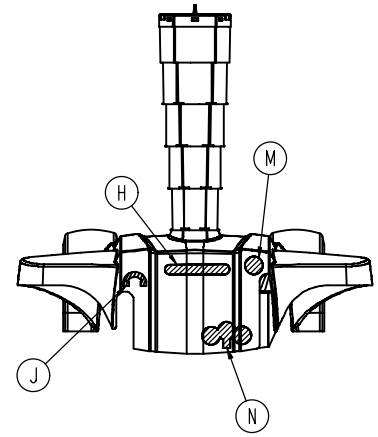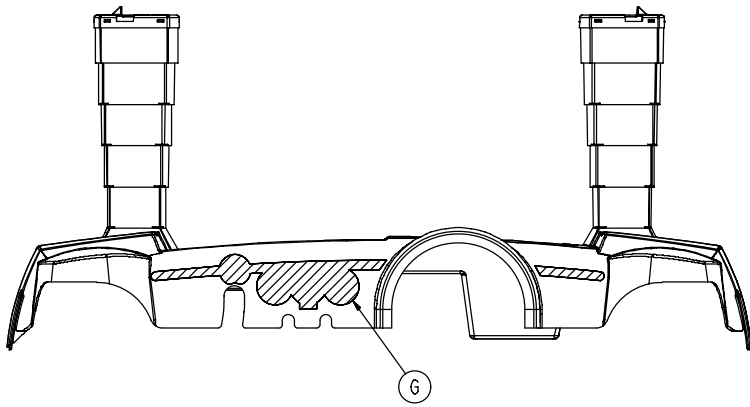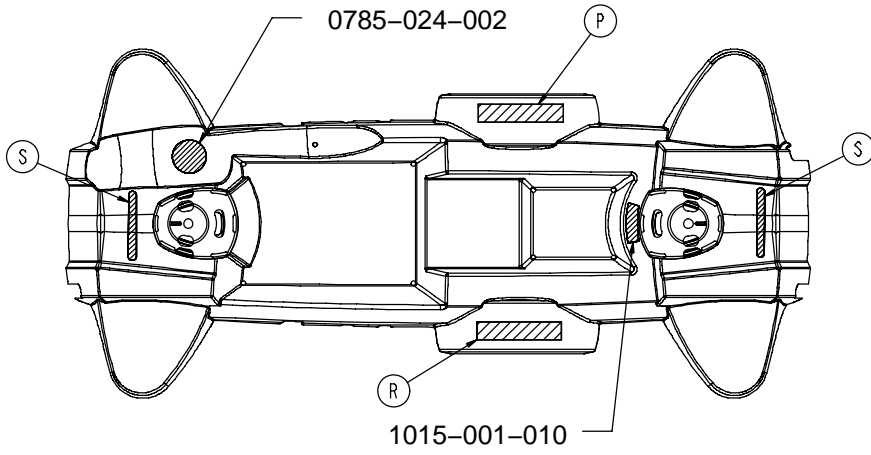

A
B

0785-001-815
1040-210-036


2
1

1015-026-400

F	G	H	J	K	L
0785-026-101	0785-026-201	0785-024-801	0785-025-600	0785-025-700	0785-022-101
0785-026-102	0785-026-202	0785-024-802	0785-025-600	0785-025-700	0785-022-102
0785-026-103	0785-026-203	0785-024-803	0785-025-600	0785-025-700	0785-022-103
0785-026-104	0785-026-204	0785-024-804	0785-025-600	0785-025-700	0785-022-104
0785-026-105	0785-026-205	0785-024-805	0785-025-600	0785-025-700	0785-022-105
0785-026-106	0785-026-206	0785-024-806	0785-025-600	0785-025-700	0785-022-106


M	N	P	R
0785-024-501	0785-024-901	0785-026-300	0785-026-400
0785-024-502	0785-024-902	0785-026-300	0785-026-400
0785-024-503	0785-024-903	0785-026-300	0785-026-400
0785-024-504	0785-024-904	0785-026-300	0785-026-400
0785-024-505	0785-024-905	0785-026-300	0785-026-400
0785-024-506	0785-024-906	0785-026-300	0785-026-400

0785-010-601

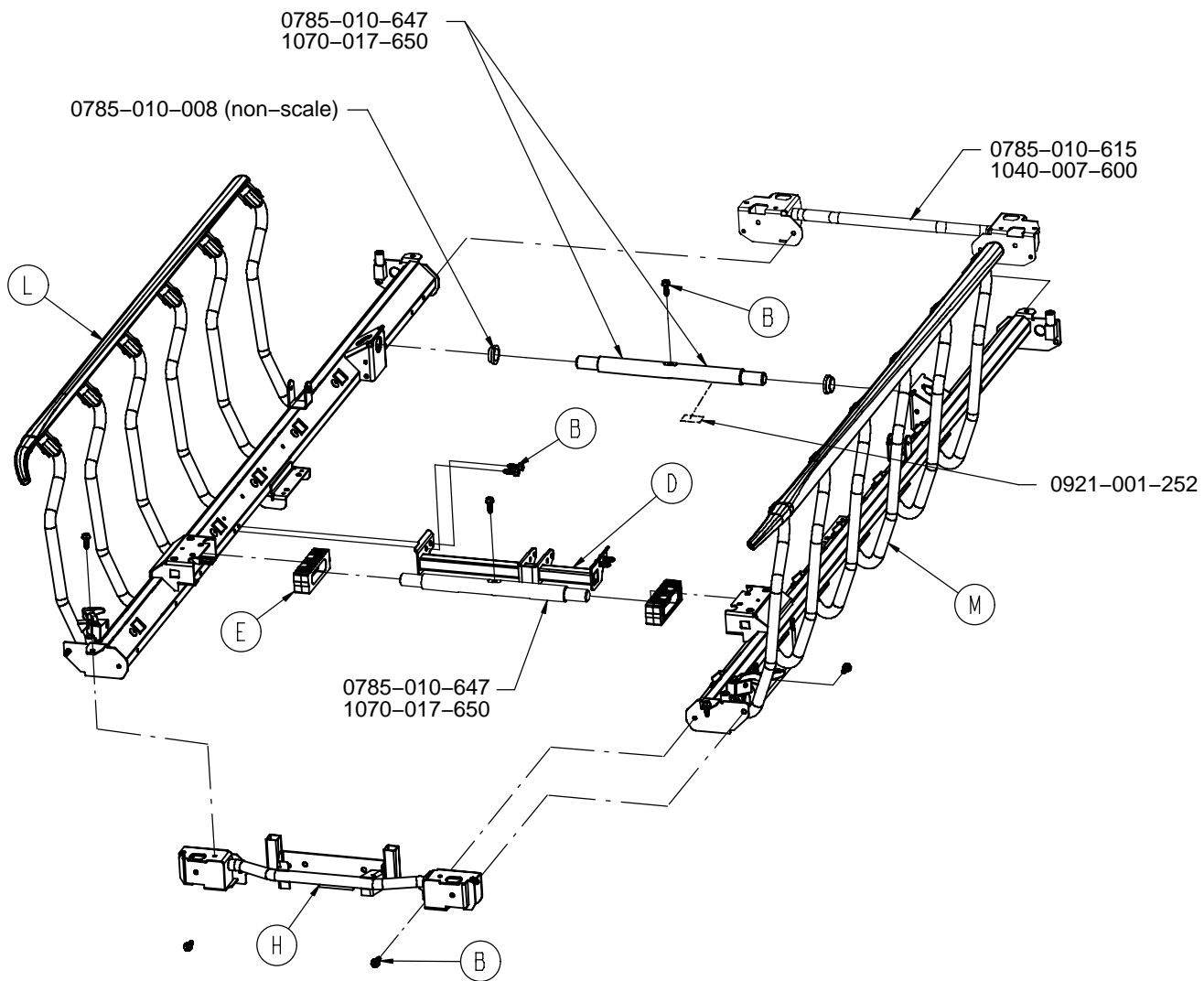


B	0023-288-000	12
D	0785-008-610	1
E	0785-010-006	2
H	0785-010-620	1
L	0785-011-040	1
M	0785-011-050	1

0785-010-301

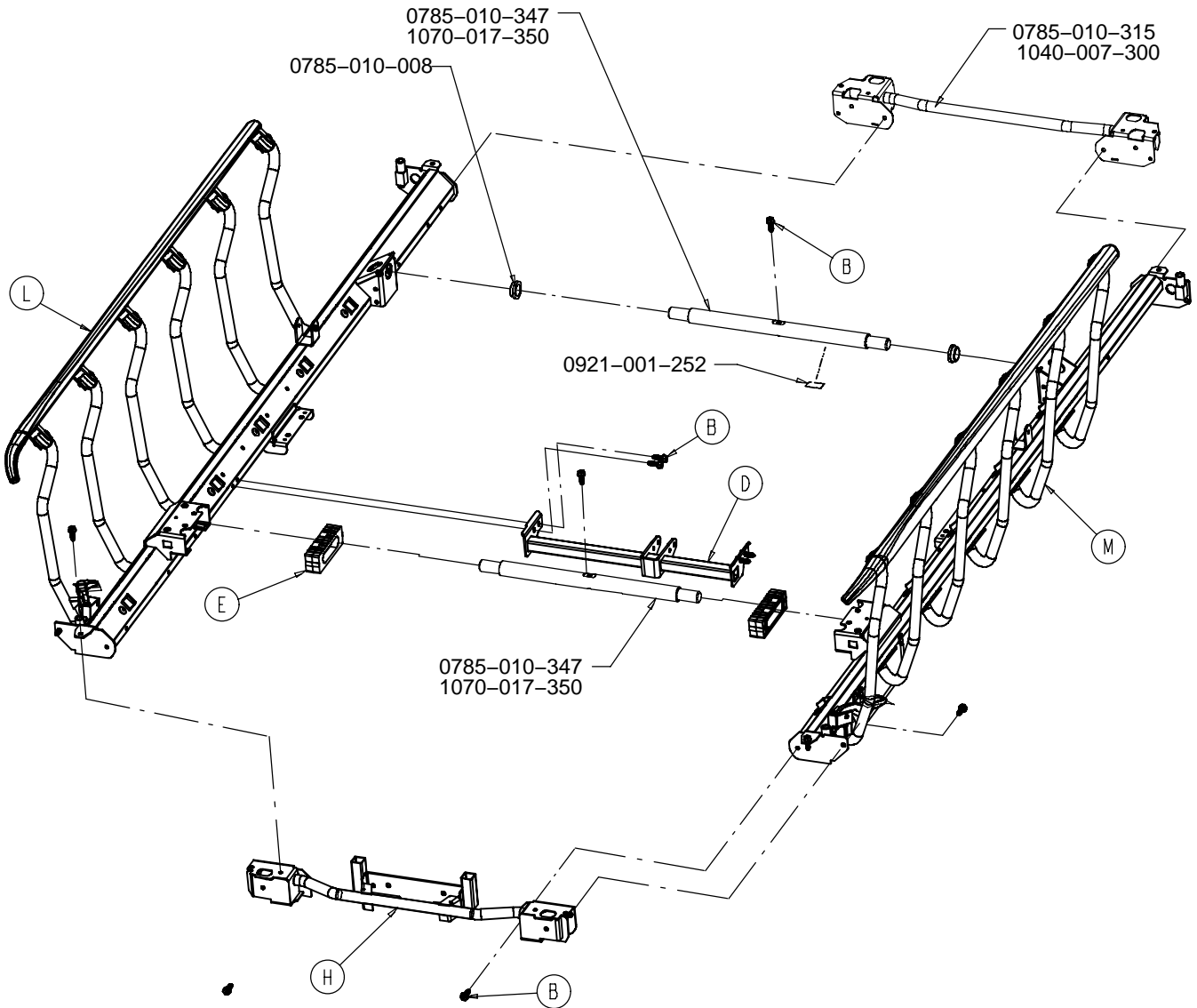


B	0023-288-000	12
D	0785-008-310	1
E	0785-010-006	2
H	0785-010-320	1
L	0785-011-040	1
M	0785-011-050	1

0785-011-040

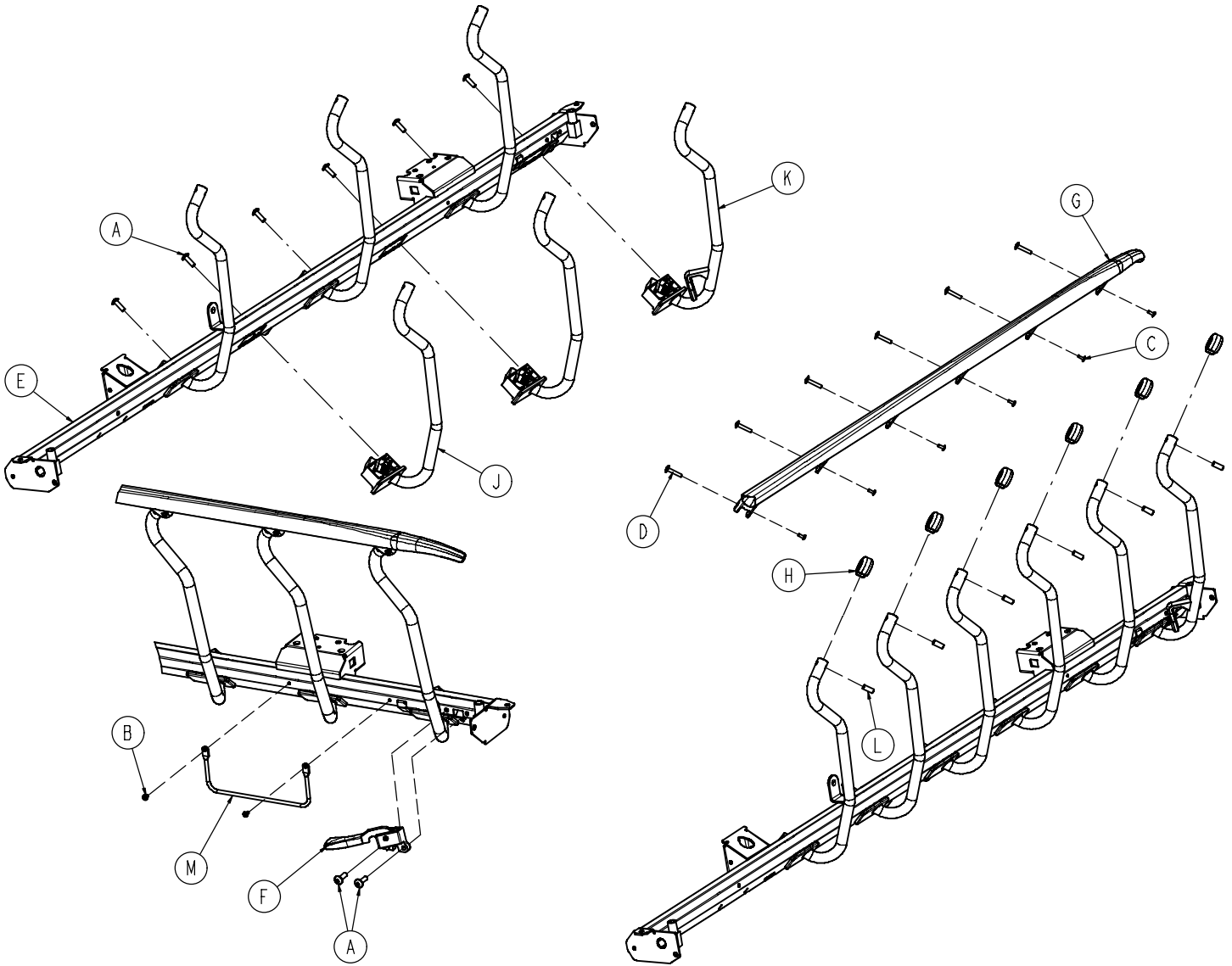


A	0007-074-000	8
B	0025-079-000	2
C	0025-182-000	6
D	0025-183-000	6
E	0785-011-012	1
F	0785-011-020	1
G	0785-011-037	1
H	0785-011-038	6
J	0785-011-041	5
K	0785-011-042	1
L	0785-011-046	6
M	1211-030-046	1

0785-011-050

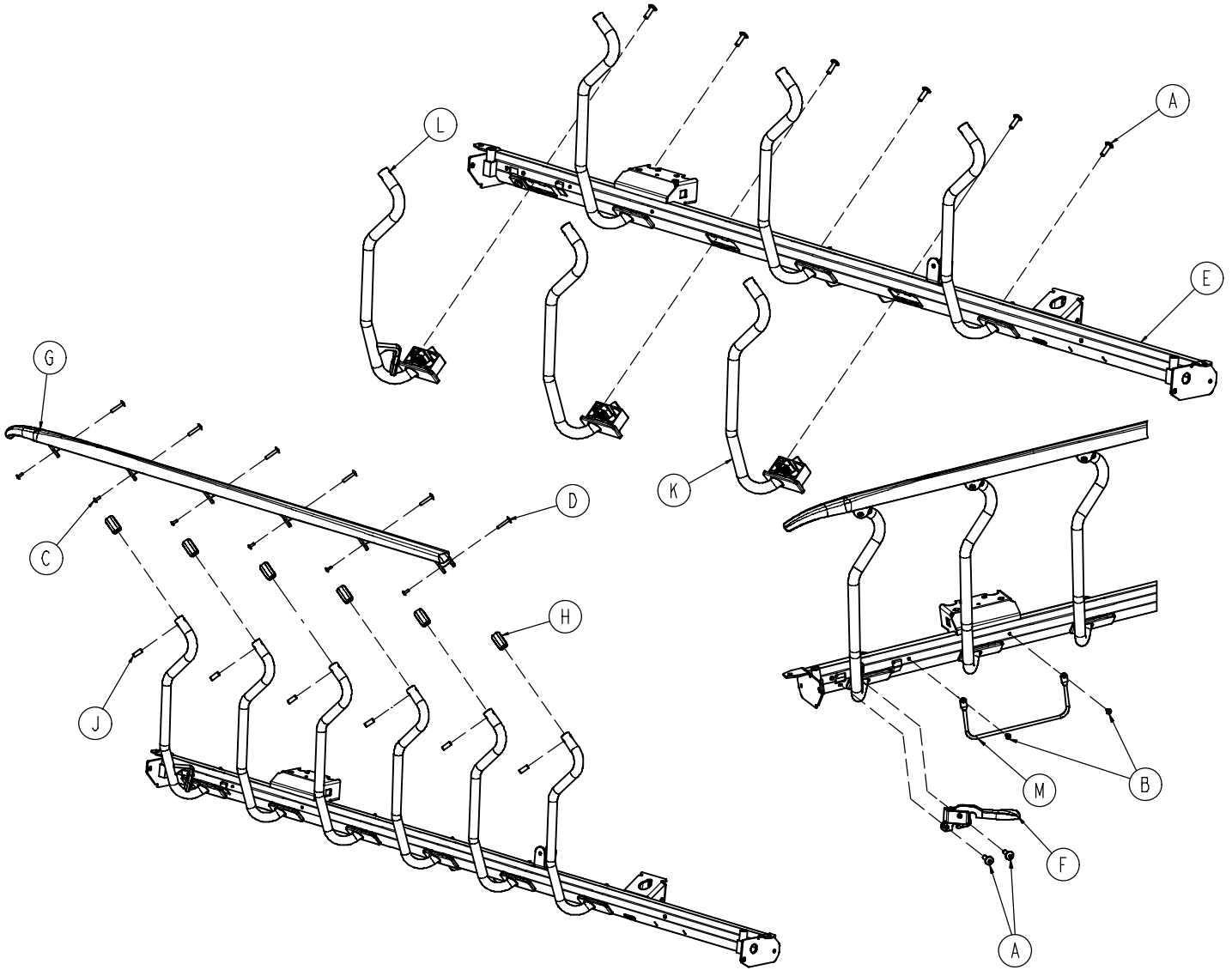


A	0007-074-000	8
B	0025-079-000	2
C	0025-182-000	6
D	0025-183-000	6
E	0785-011-011	1
F	0785-011-021	1
G	0785-011-036	1
H	0785-011-038	6
J	0785-011-046	6
K	0785-011-051	5
L	0785-011-052	1
M	1211-030-046	1

0785-011-020

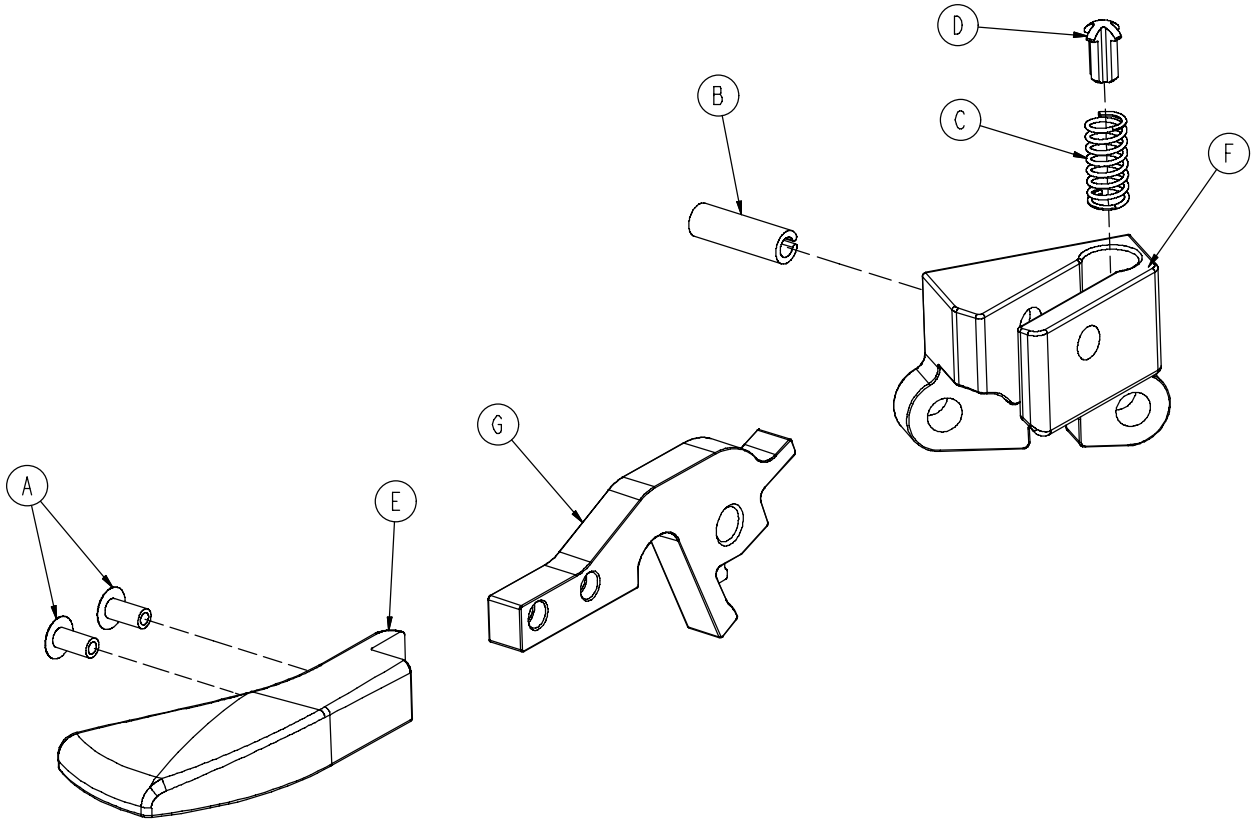


A	0025-079-000	2
B	0026-365-000	1
C	0038-526-000	1
D	0785-011-054	1
E	0785-011-056	1
F	0785-011-128	1
G	0785-011-131	1

0785-011-021

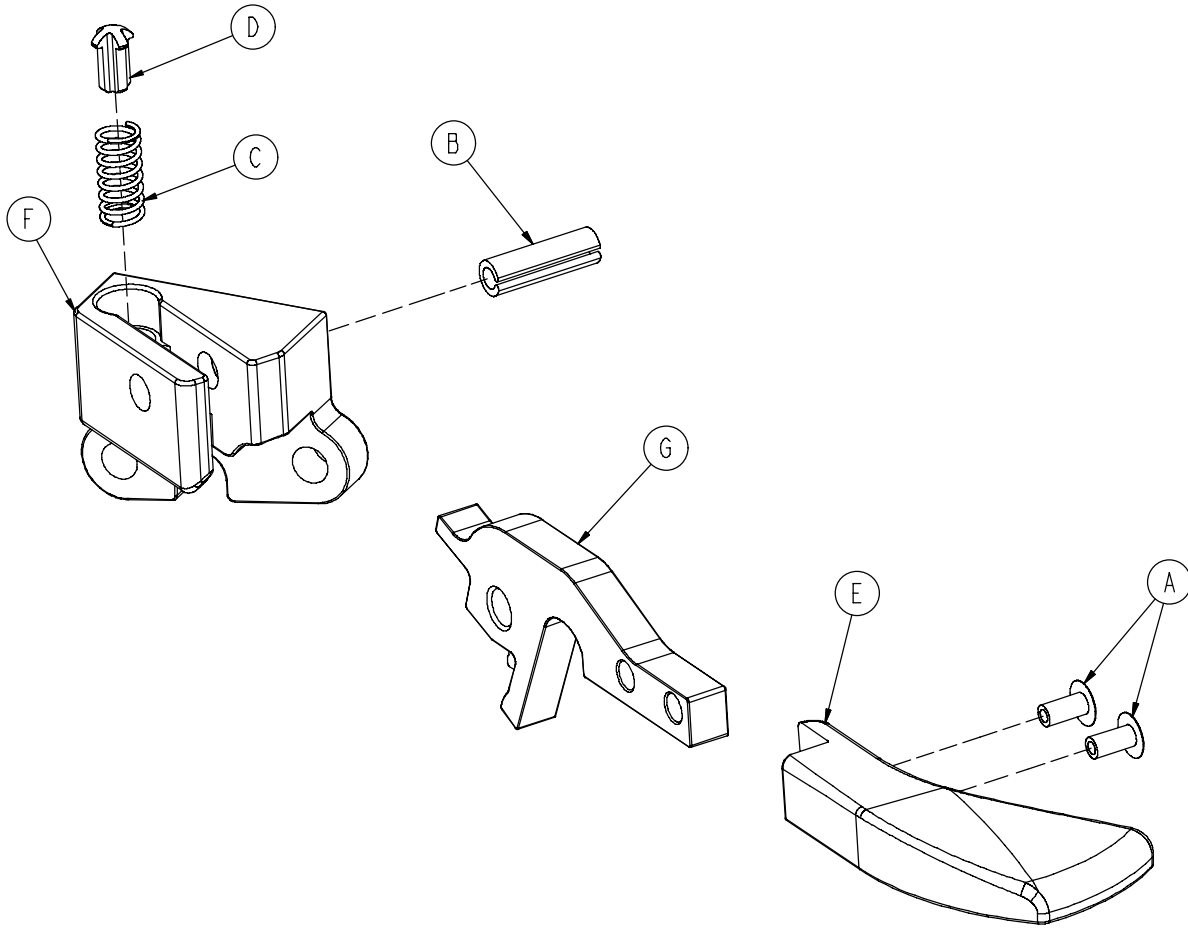


A	0025-079-000	2
B	0026-365-000	1
C	0038-526-000	1
D	0785-011-054	1
E	0785-011-057	1
F	0785-011-123	1
G	0785-011-131	1

0785-011-041

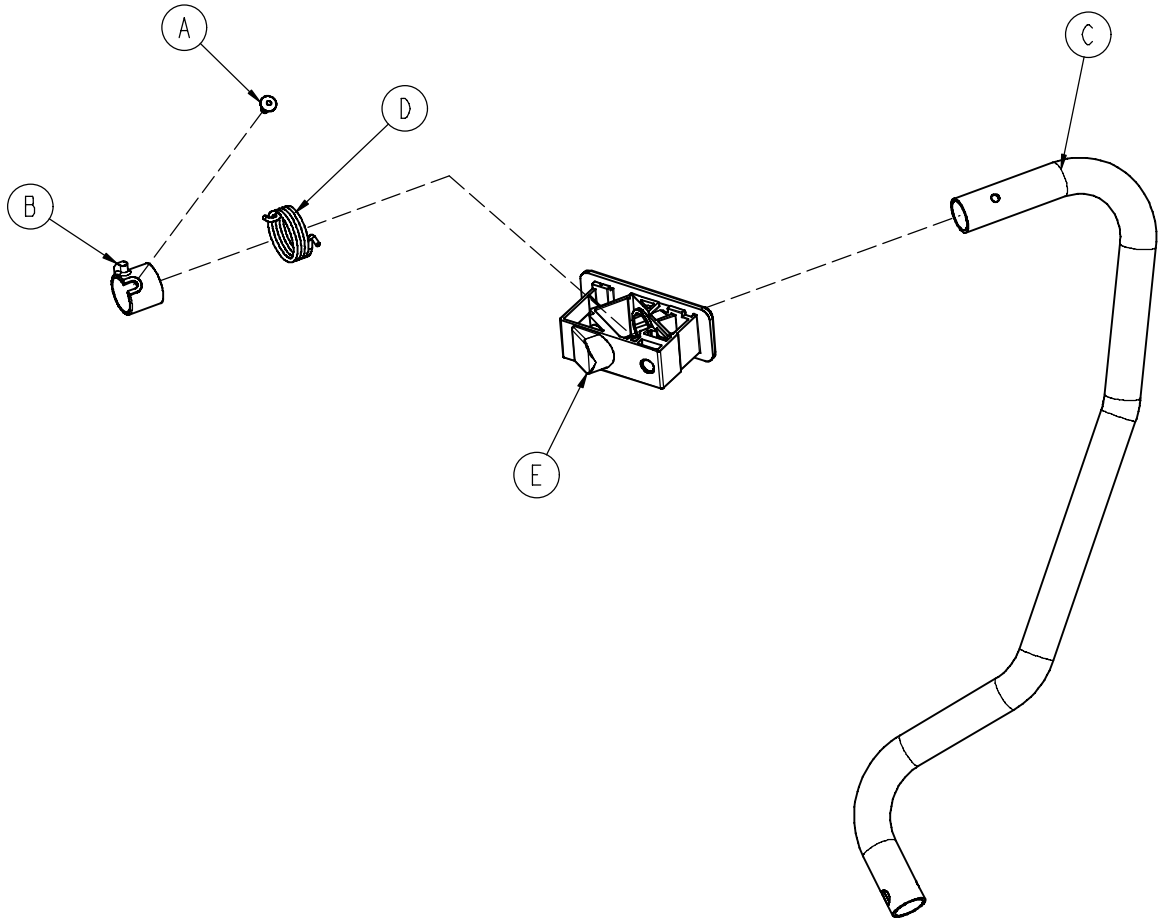


A	0003-099-000	1
A	0025-079-000	1
B	0785-011-013	1
C	0785-011-032	1
D	0785-011-047	1
E	0785-011-197	1

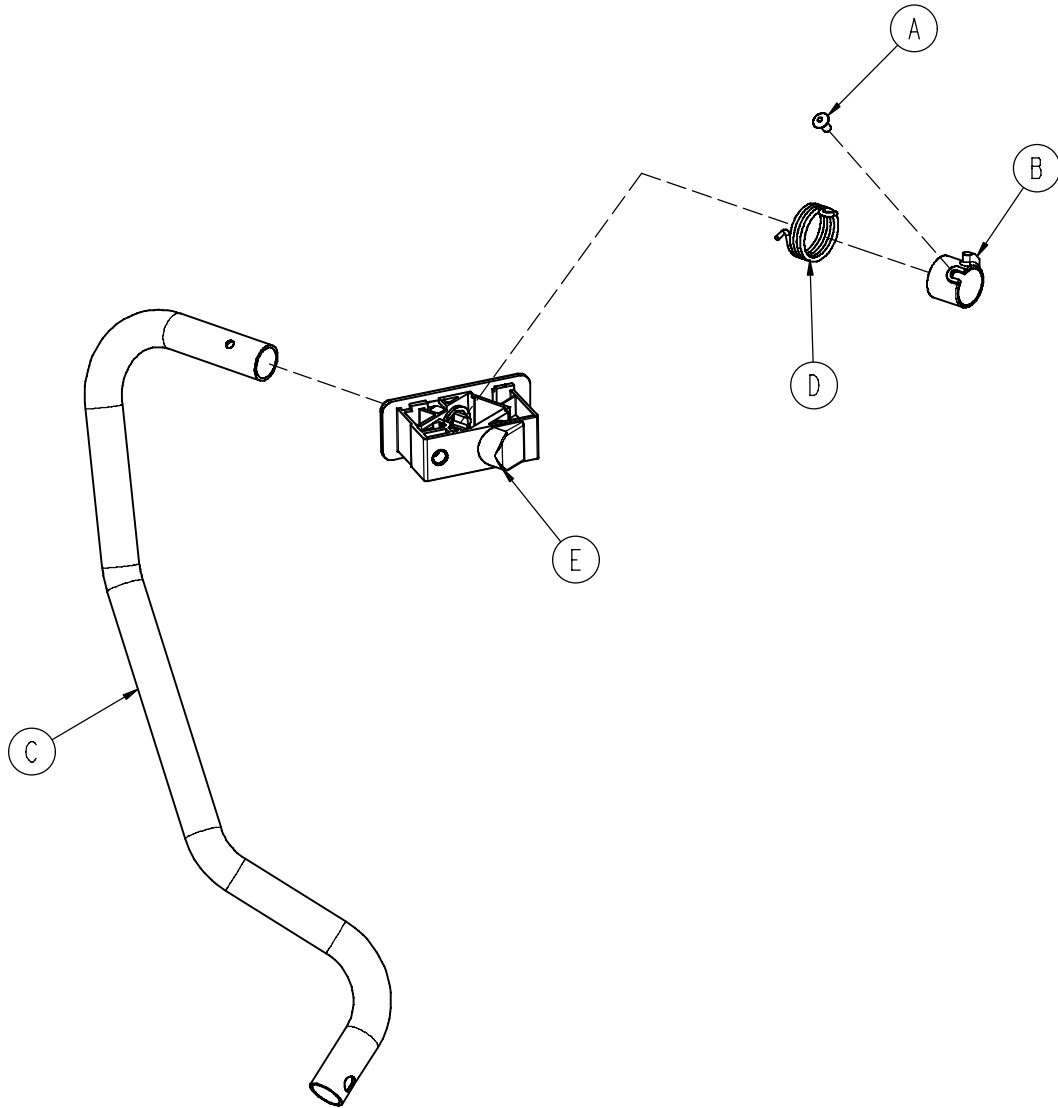


A	0025-079-000	1
B	0785-011-010	1
C	0785-011-033	1
D	0785-011-048	1
E	0785-011-197	1

0785-011-042

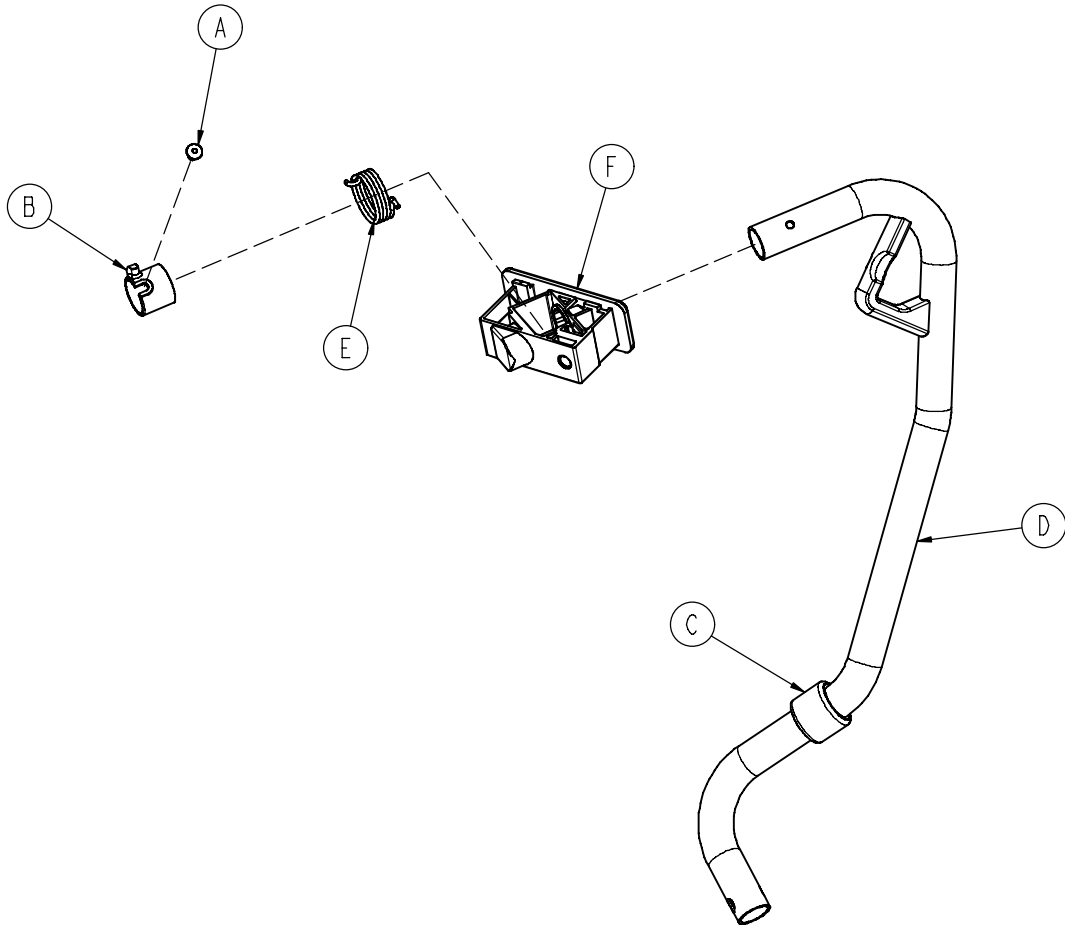


A	0025-079-000	1
B	0785-011-013	1
C	0785-011-016	1
D	0785-011-043	1
E	0785-011-047	1
F	0785-011-197	1

0785-011-052

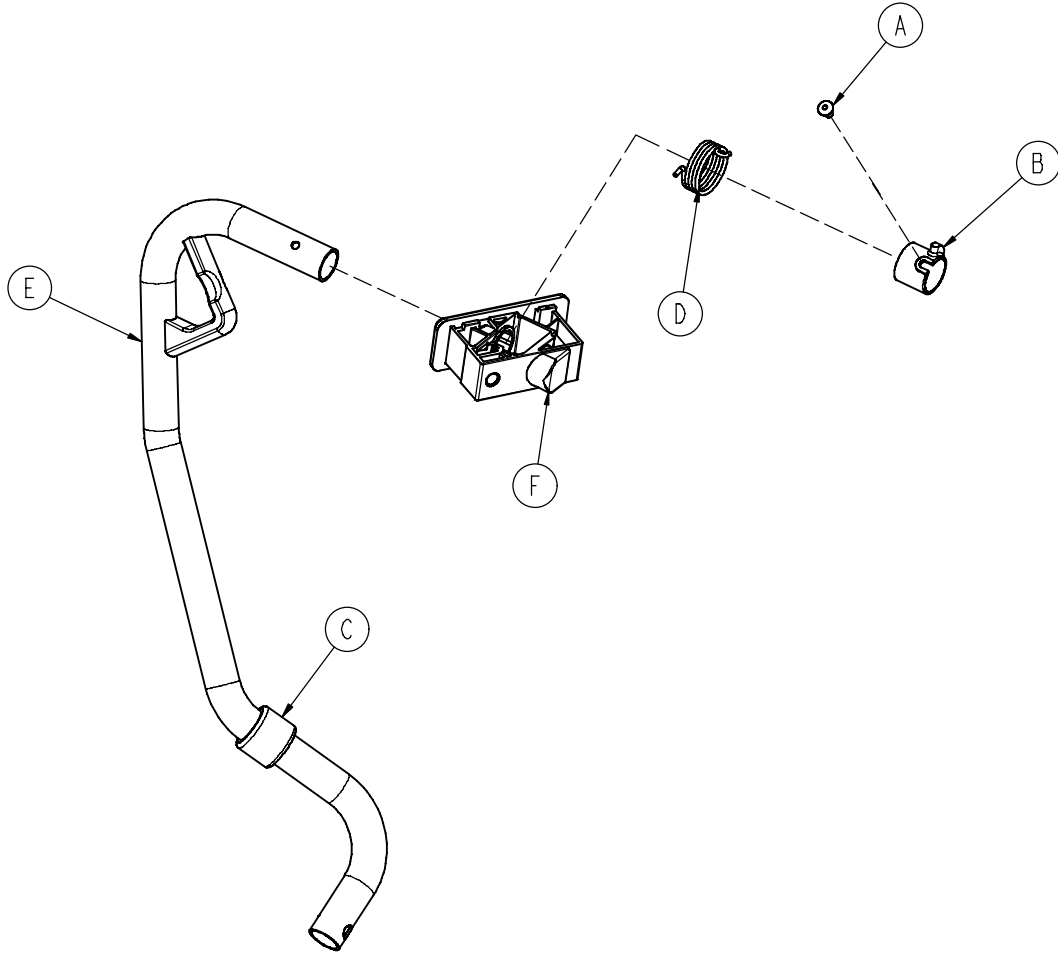


A	0025-079-000	1
B	0785-011-010	1
C	0785-011-016	1
D	0785-011-048	1
E	0785-011-053	1
F	0785-011-197	1

0785-011-140

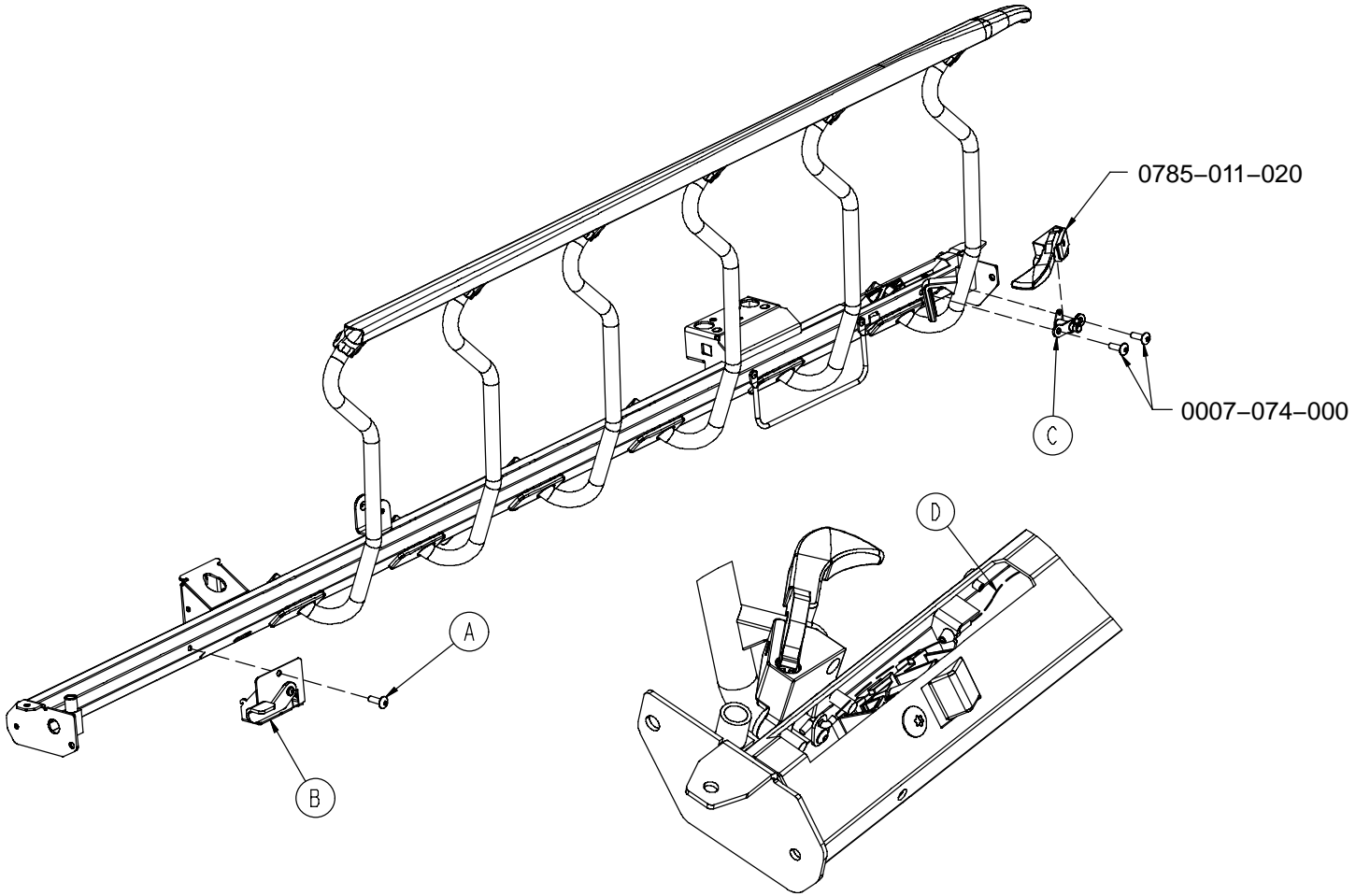


A	0007-074-000	1
B	0785-011-070	1
C	0785-011-100	1
D	0785-011-110	1

0785-011-150

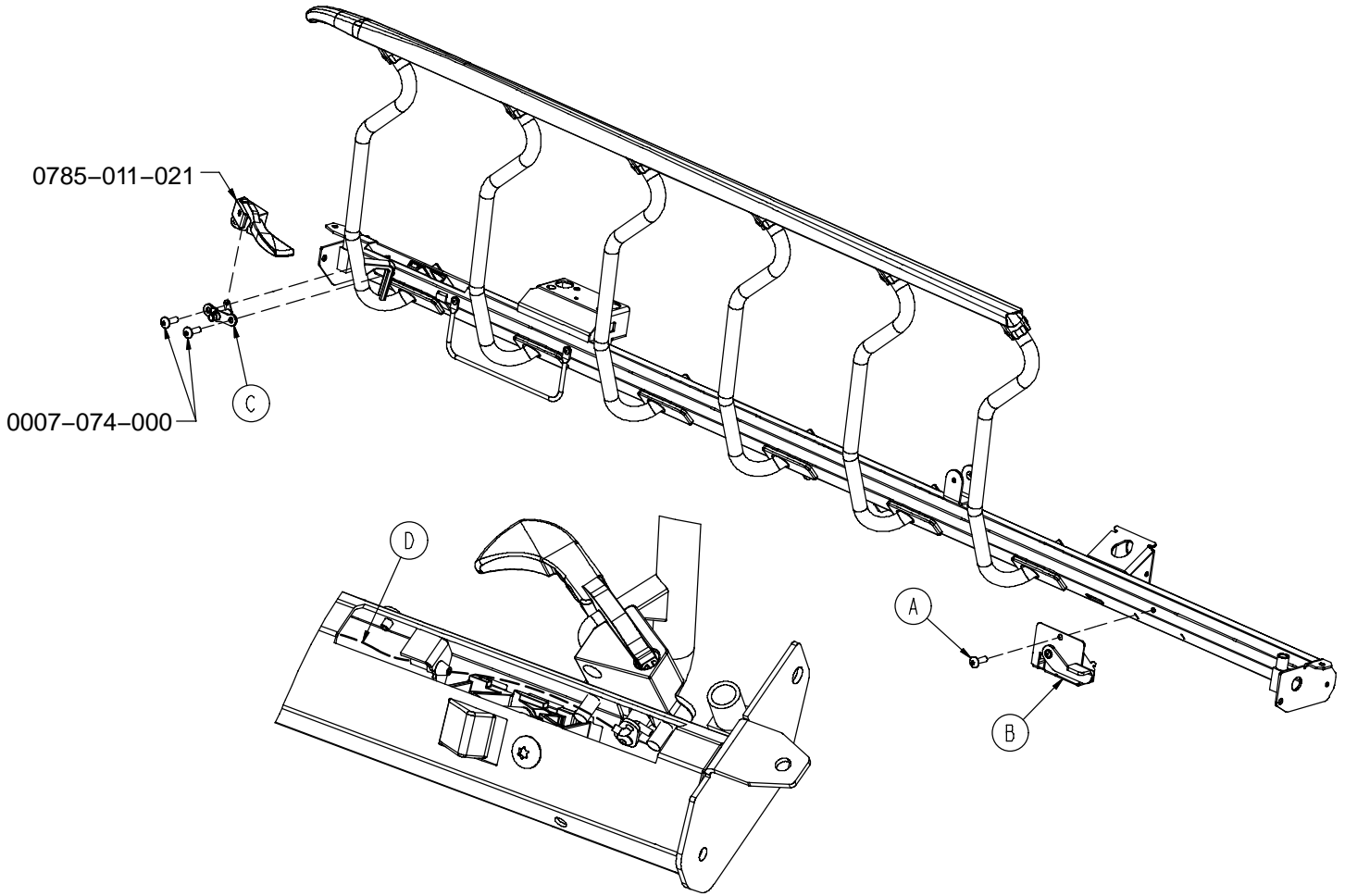


A	0007-074-000	1
B	0785-011-130	1
C	0785-011-135	1
D	0785-011-110	1

0785-011-070

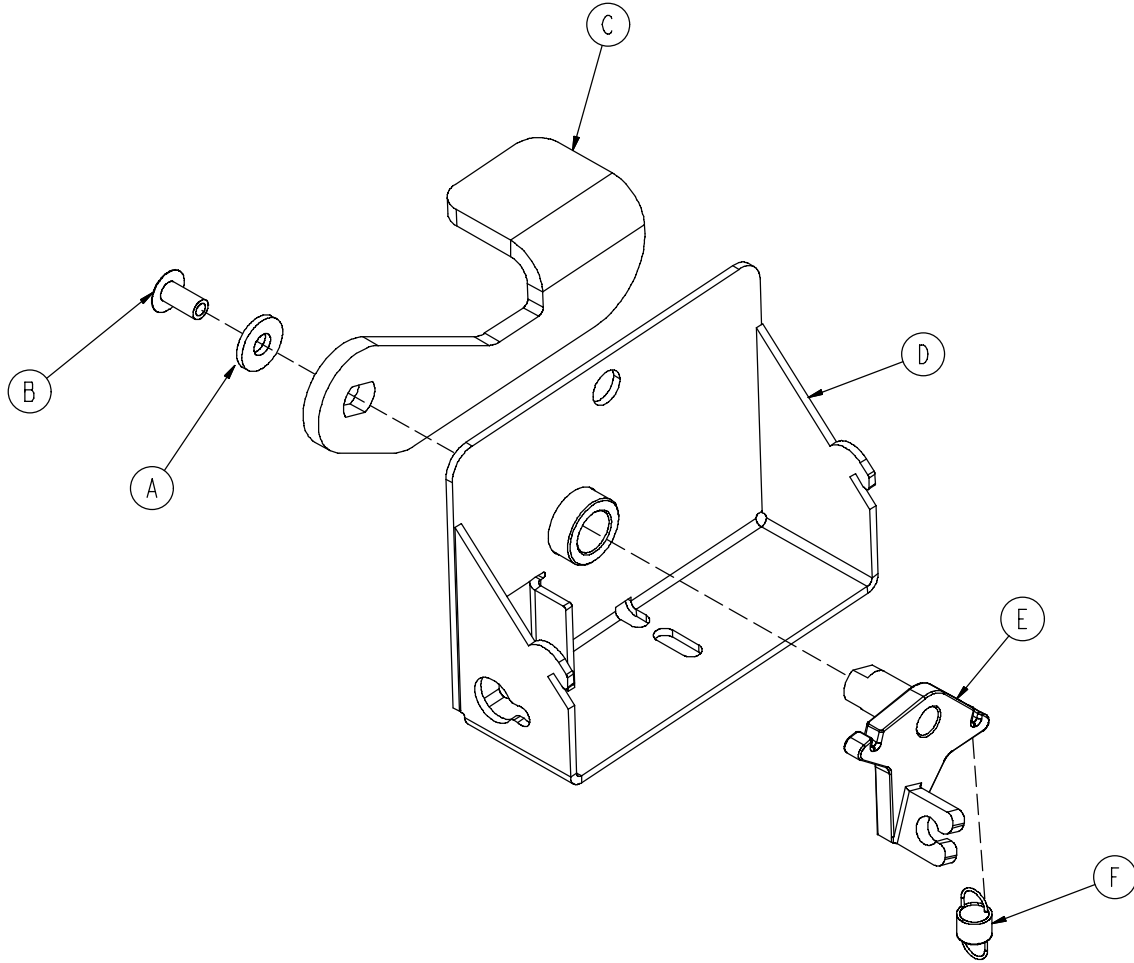


A	0011-063-000	1
B	0025-079-000	1
C	0785-011-074	1
D	0785-011-075	1
E	0785-011-077	1
F	0785-011-093	1

0785-011-100

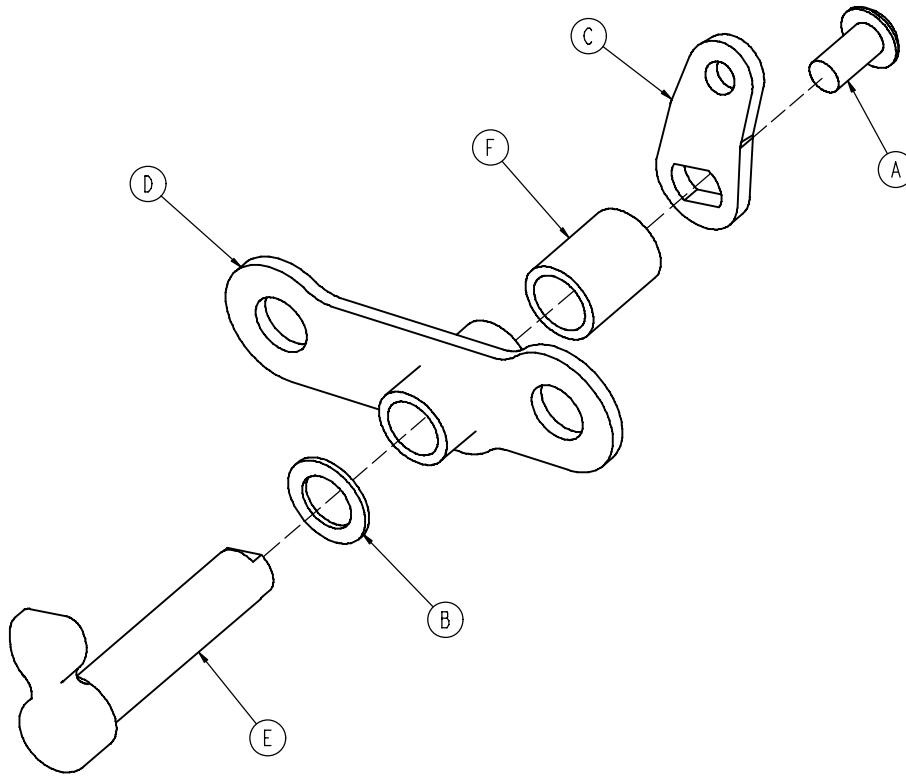


A	0004-136-000	1
B	0014-122-000	1
C	0785-011-086	1
D	0785-011-090	1
E	0785-011-095	1
F	0785-011-096	1

0785-011-070

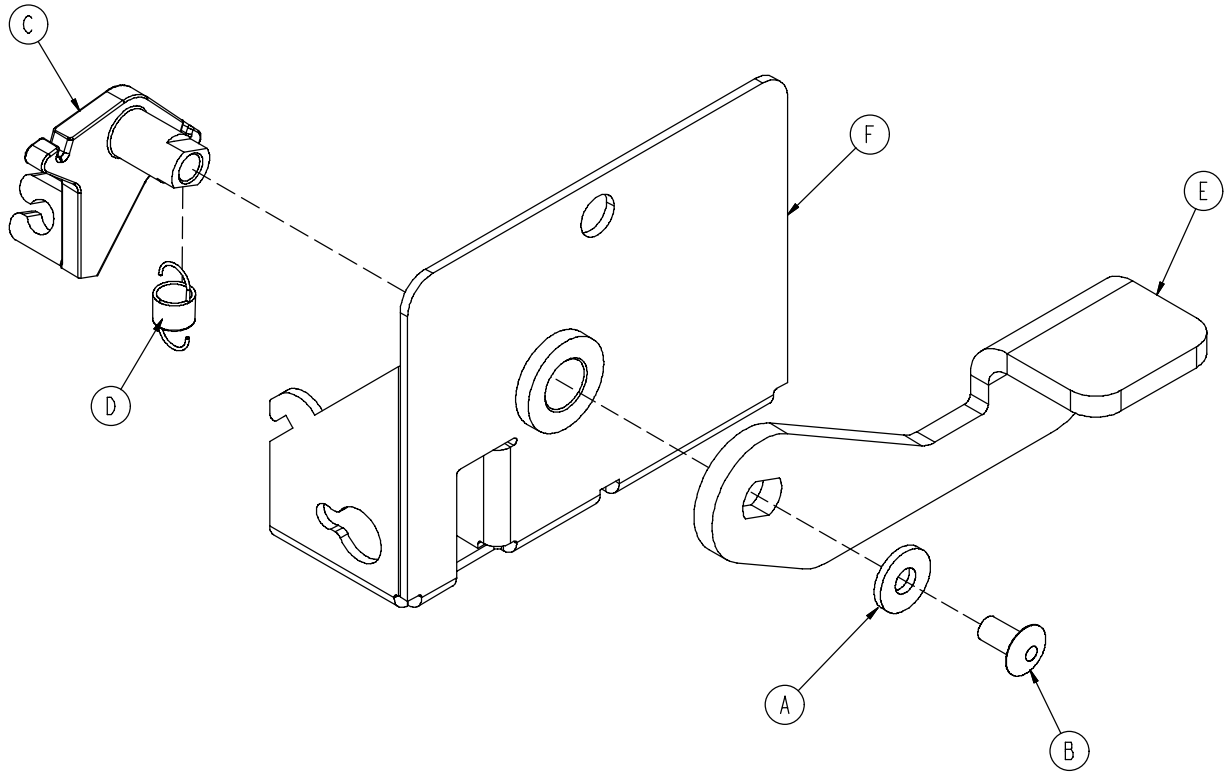


A	0011-063-000	1
B	0025-079-000	1
C	0785-011-077	1
D	0785-011-093	1
E	0785-011-106	1
F	0785-011-120	1

0785-011-135

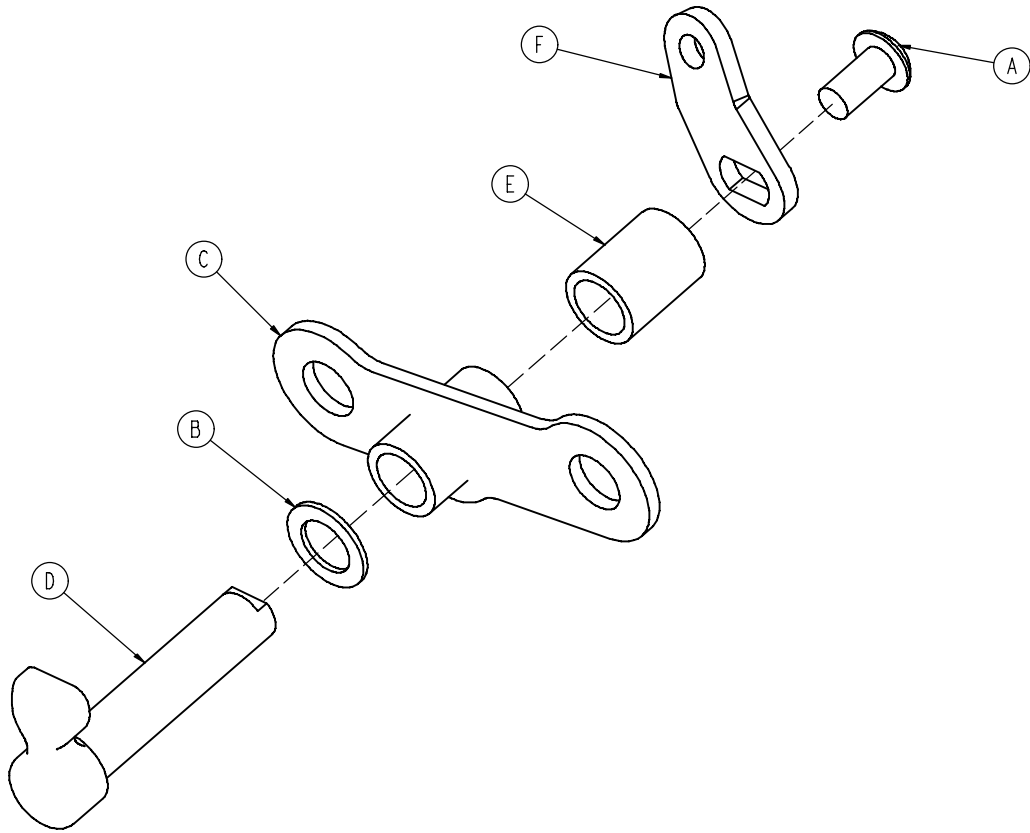


A	0004-036-000	1
B	0014-122-000	1
C	0785-011-090	1
D	0785-011-095	1
E	0785-011-096	1
F	0785-011-186	1

0785-010-606

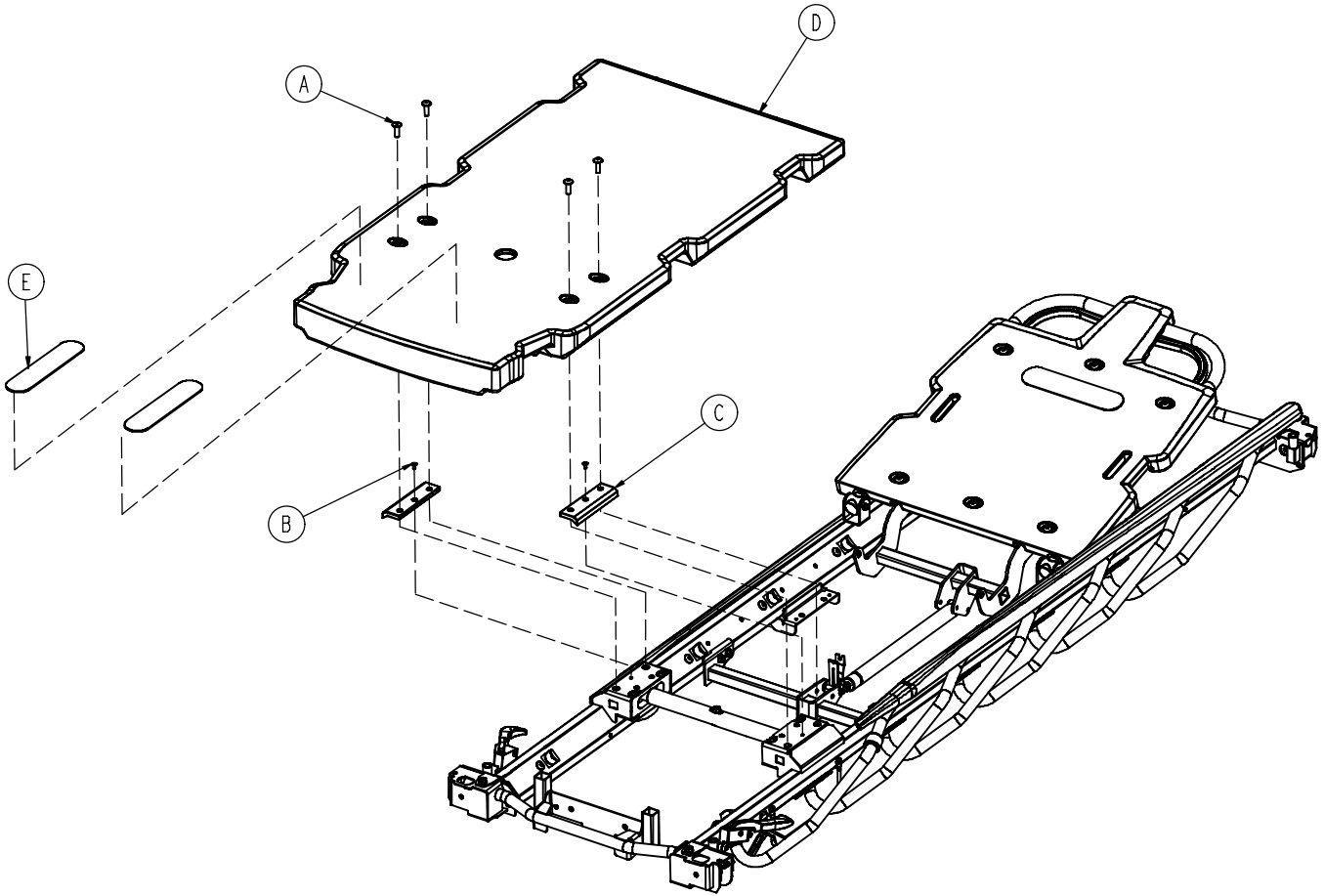


A	0007-074-000	4
B	0025-079-000	2
C	0785-012-107	2
D	0785-015-650	1
E	0785-034-005	2


0785-010-306

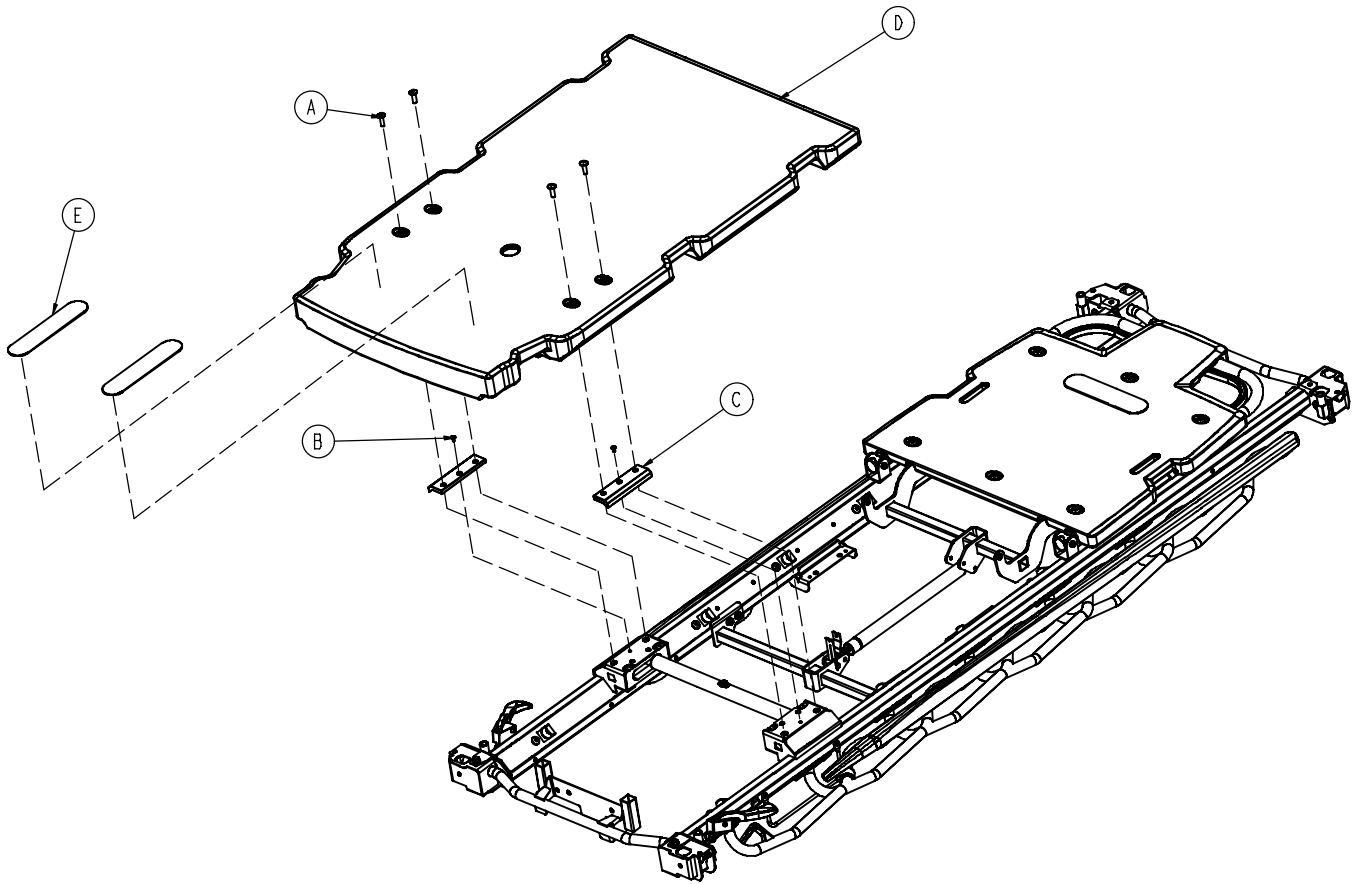


A	0007-074-000	4
B	0025-079-000	2
C	0785-012-107	2
D	0785-015-350	1
E	0785-034-005	2

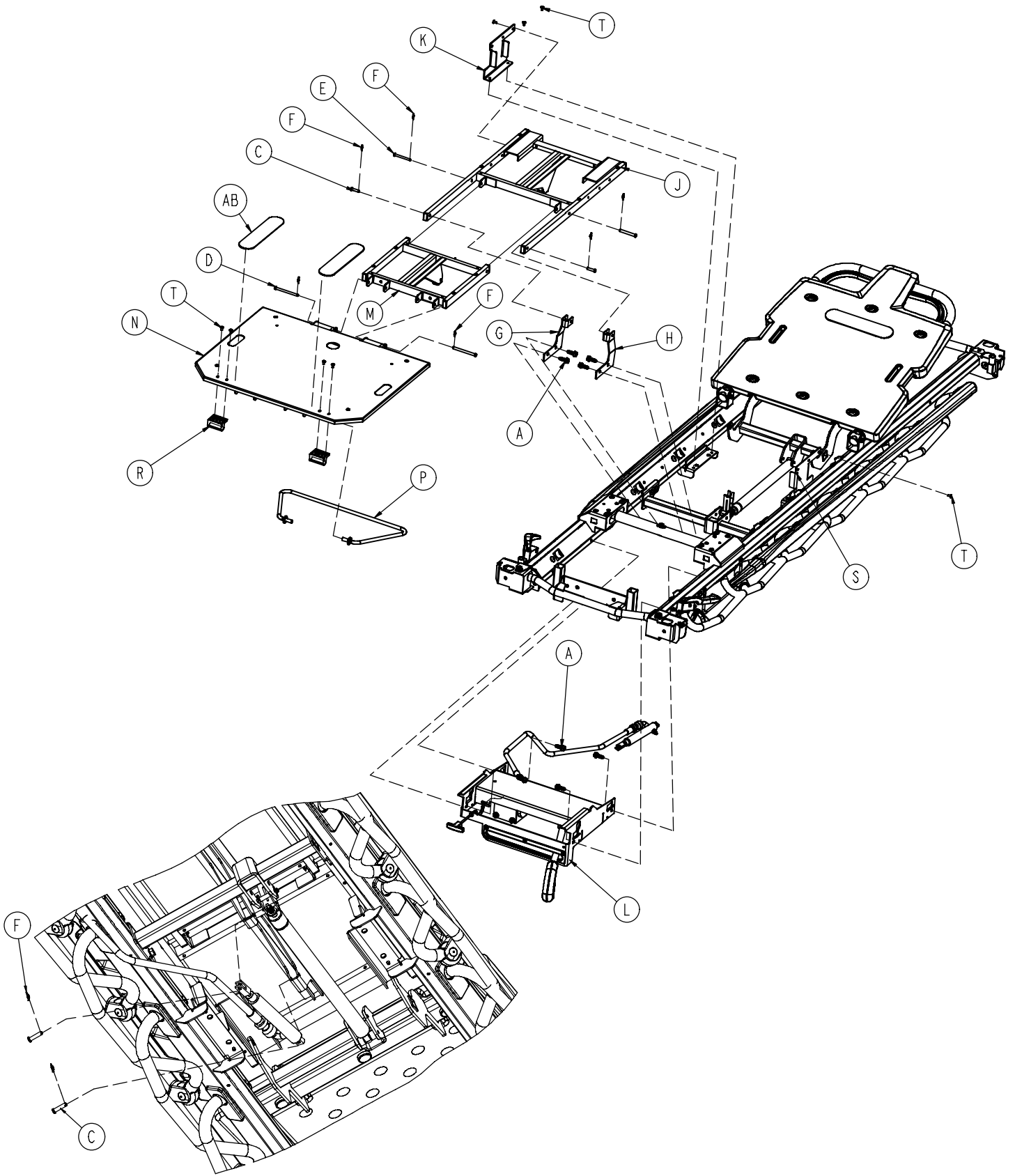


0785-010-603

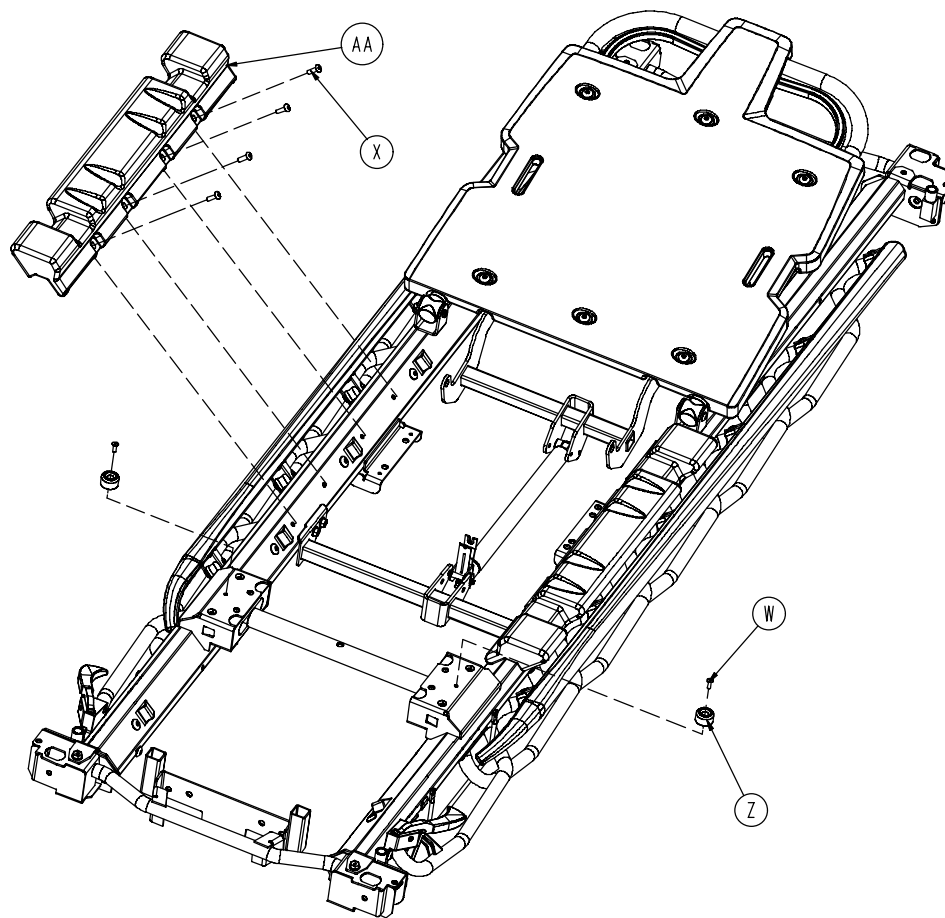


A	0023-288-000	8
C	0026-316-000	4
D	0026-364-000	2
E	0026-334-000	2
F	0027-020-000	8
G	0785-013-034	1
H	0785-013-039	1
J	0785-013-610	1
K	0785-014-119	1
L	0785-014-602	1
M	0785-014-610	1
N	0785-015-601	1
P	0785-015-609	1
R	0785-015-012	2
S	0785-014-117	1
T	0025-079-000	10
W	0025-133-000	2
X	0025-147-000	8
Z	0056-023-000	2
AA	0785-010-049	2
AB	0785-034-005	2

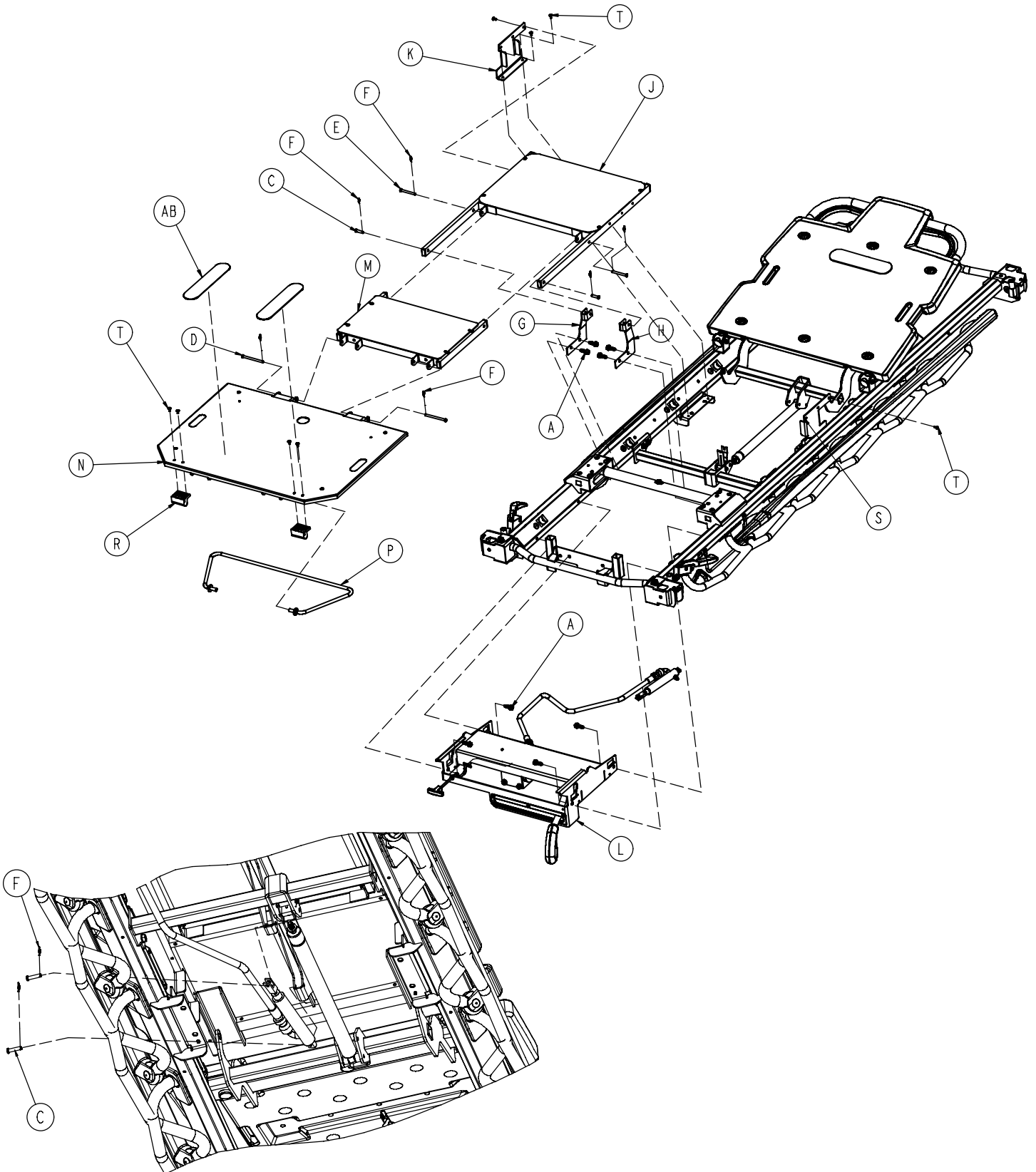


0785-010-303

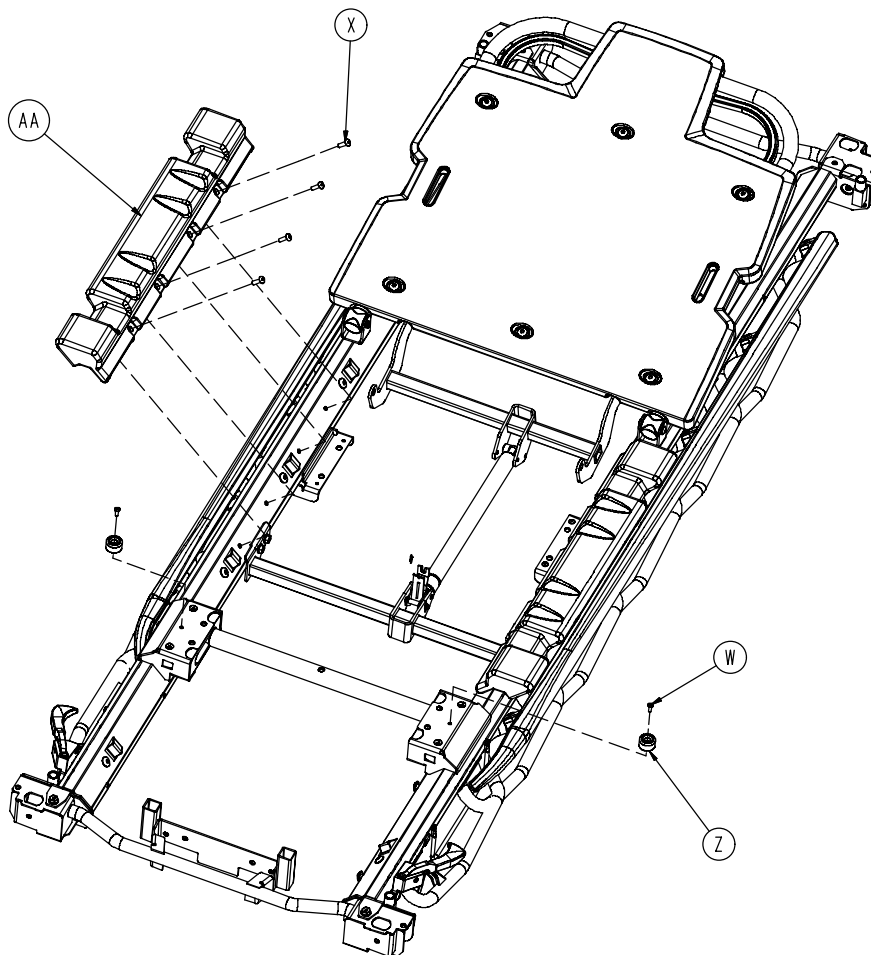

A	0023-288-000	8
C	0026-316-000	4
D	0026-364-000	2
E	0026-334-000	2
F	0027-020-000	8
G	0785-013-034	1
H	0785-013-039	1
J	0785-013-310	1
K	0785-014-119	1
L	0785-014-302	1
M	0785-014-310	1
N	0785-015-301	1
P	0785-015-309	1
R	0785-015-012	2
S	0785-014-117	1
T	0025-079-000	10
W	0025-133-000	2
X	0025-147-000	8
Z	0056-023-000	2
AA	0785-010-049	2
AB	0785-034-005	2

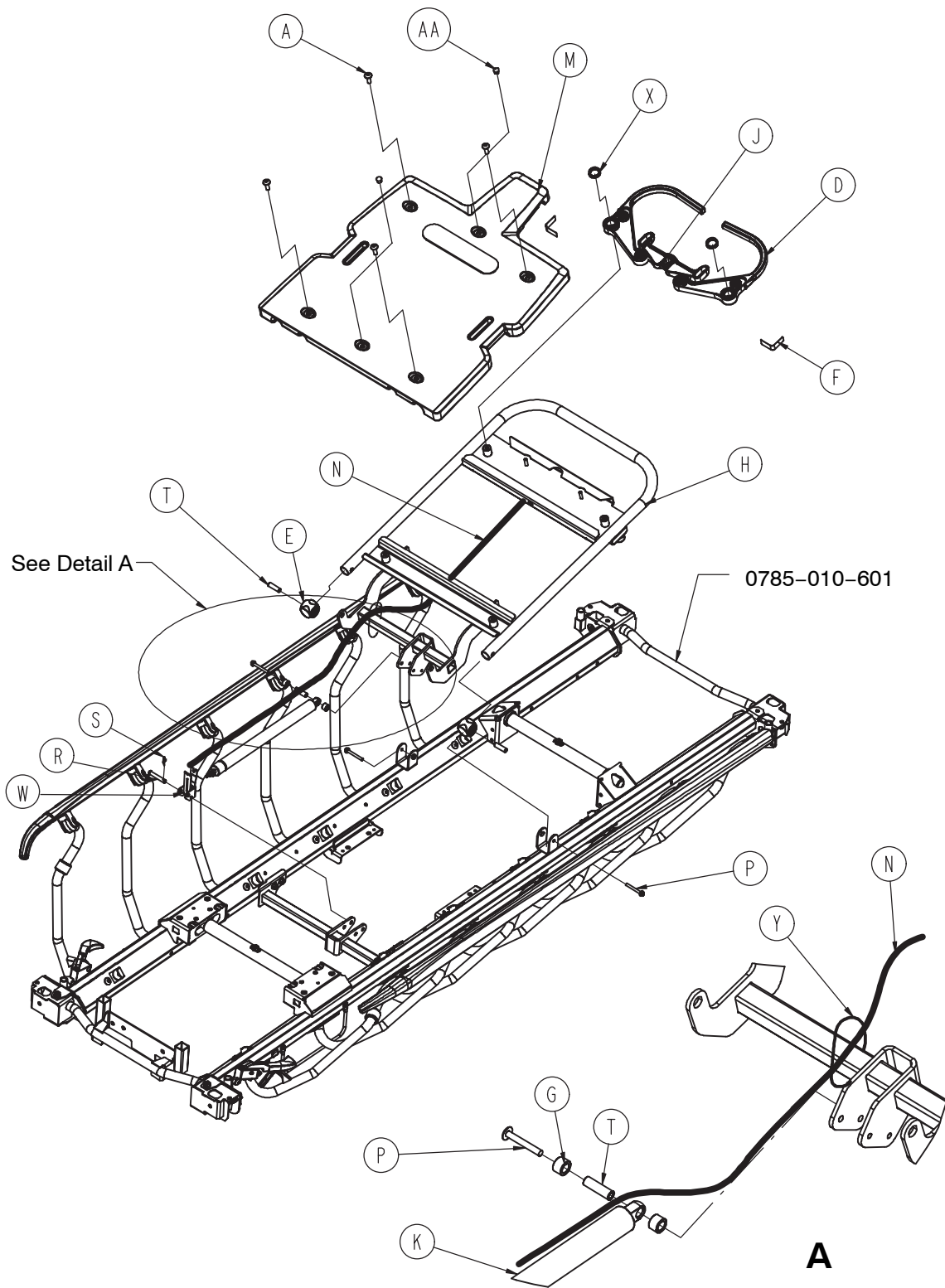


0785-012-602

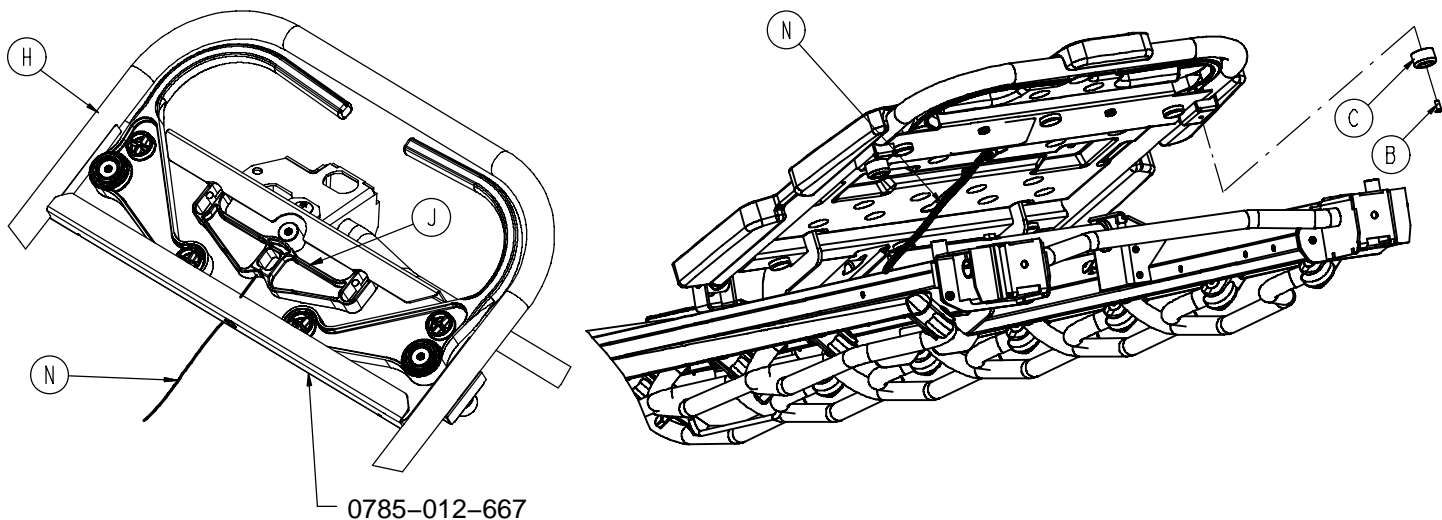


A	0007-074-000	4
B	0025-079-000	2
C	0056-023-000	2
D	0785-012-066	2
E	0785-012-076	2
F	0785-012-086	2
G	0785-012-094	2
H	0785-012-605	1
J	0785-012-666	1
K	0360-031-077	1
M	0785-012-650	1
N	0785-012-080	1
P	0003-385-000	3
R	0026-296-000	1
S	0027-019-000	1
T	0785-012-103	3
W	0785-012-116	1
X	0038-557-000	2
Y	8815-029-200	1
AA	0037-065-000	2

0785-012-302

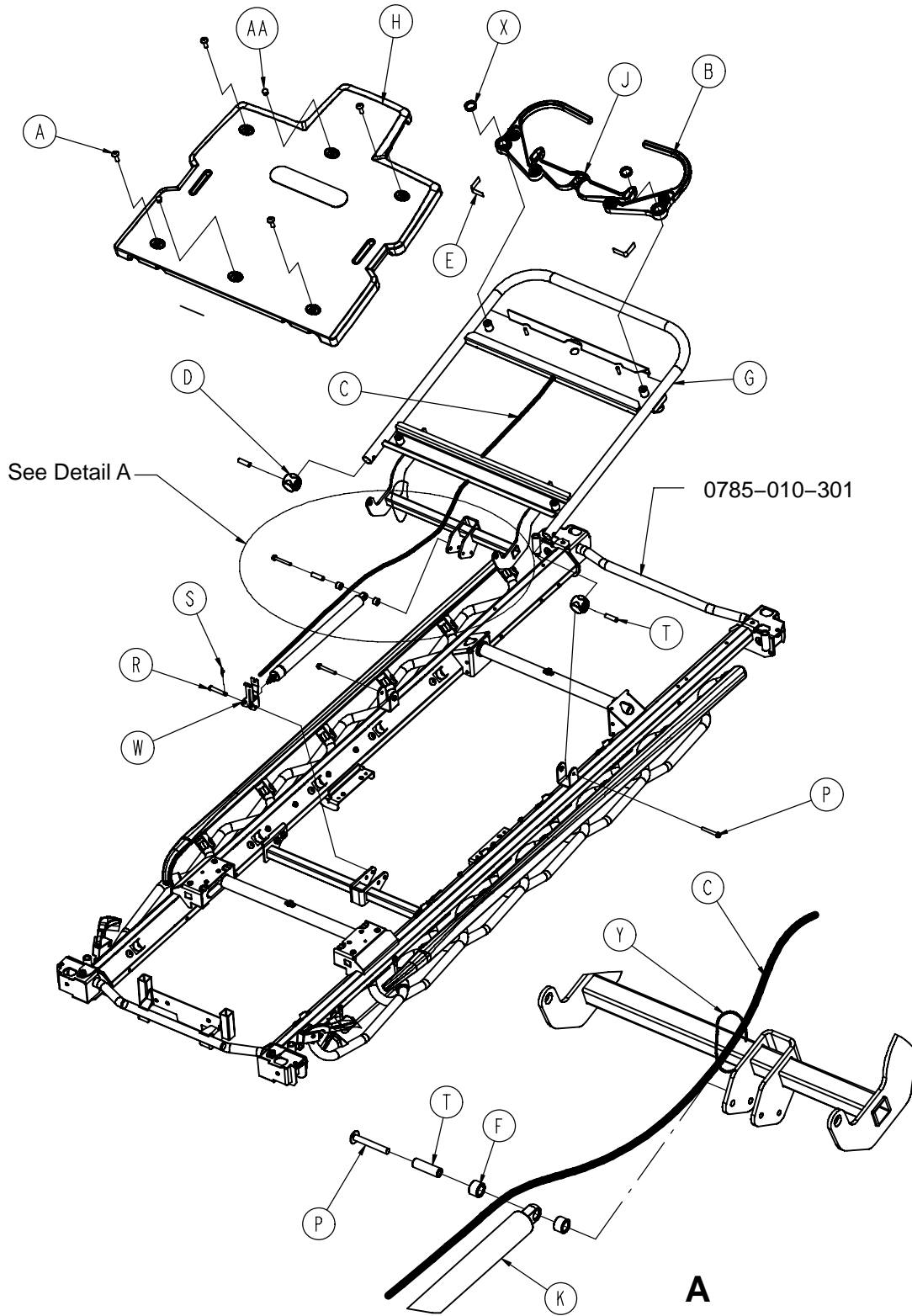


0785-012-302

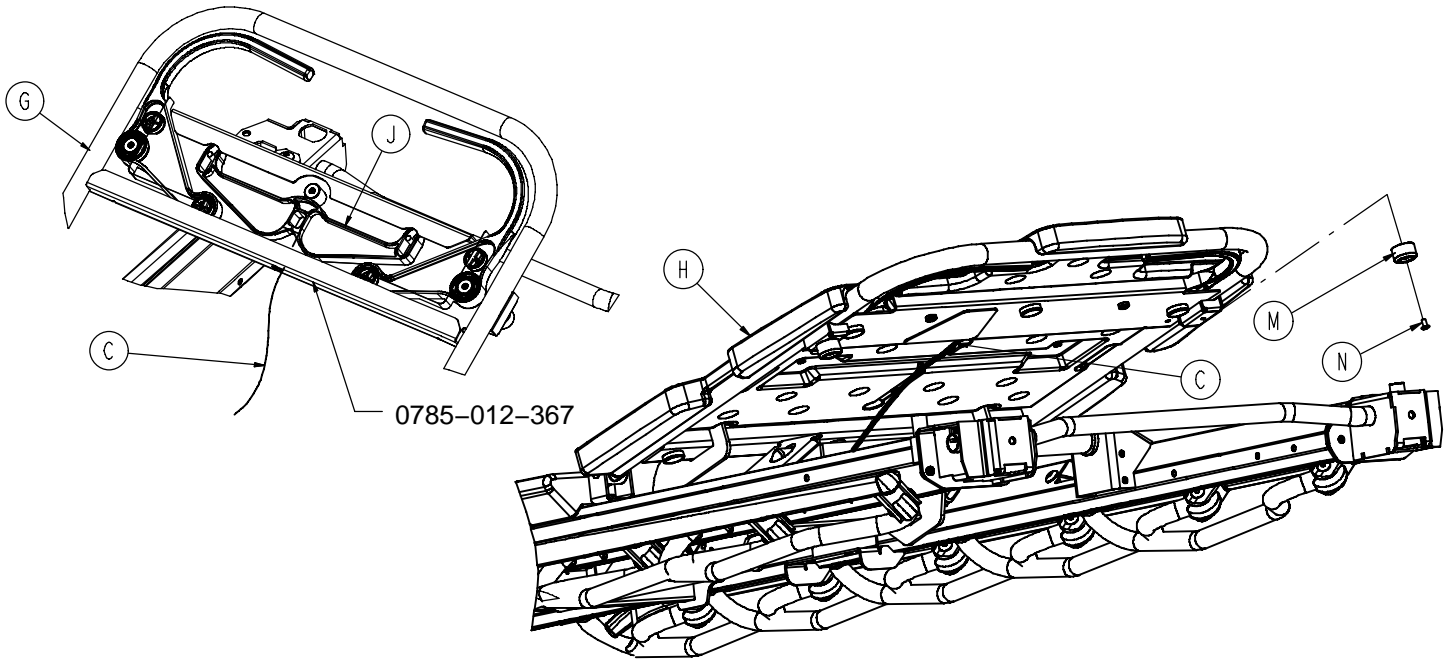


A	0007-074-000	4
B	0785-012-066	2
C	0785-012-080	1
D	0785-012-076	2
E	0785-012-086	2
F	0785-012-094	2
G	0785-012-305	1
H	0785-012-350	1
J	0785-012-366	1
K	0360-031-077	1
M	0056-023-000	2
N	0025-079-000	2
P	0003-385-000	3
R	0026-296-000	1
S	0027-019-000	1
T	0785-012-103	3
W	0785-012-116	1
X	0038-557-000	2
Y	8815-029-200	1
AA	0037-065-000	2

0785-013-610

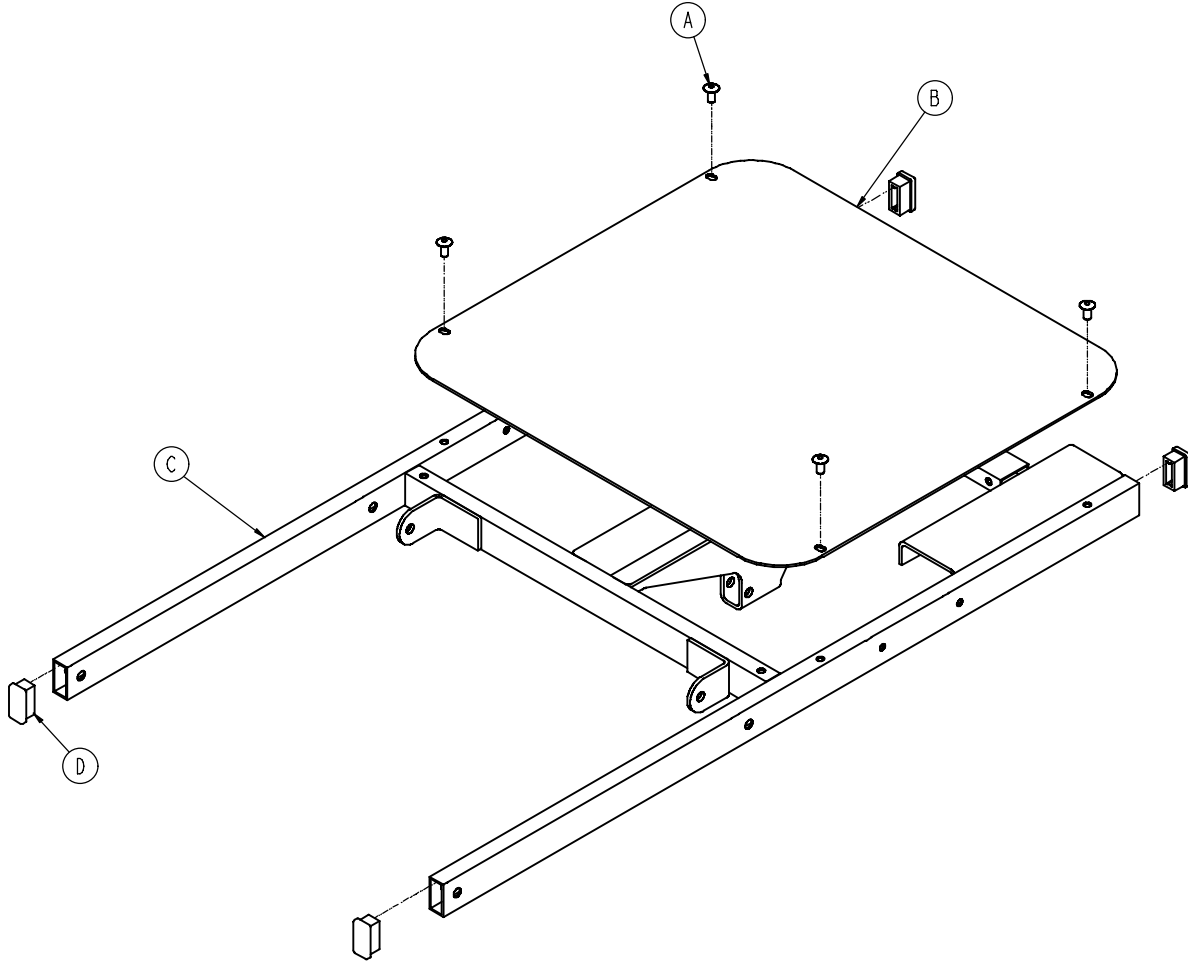

A	0025-079-000	4
B	0785-013-619	1
C	0785-013-625	1
D	1010-032-066	4


0785-013-310

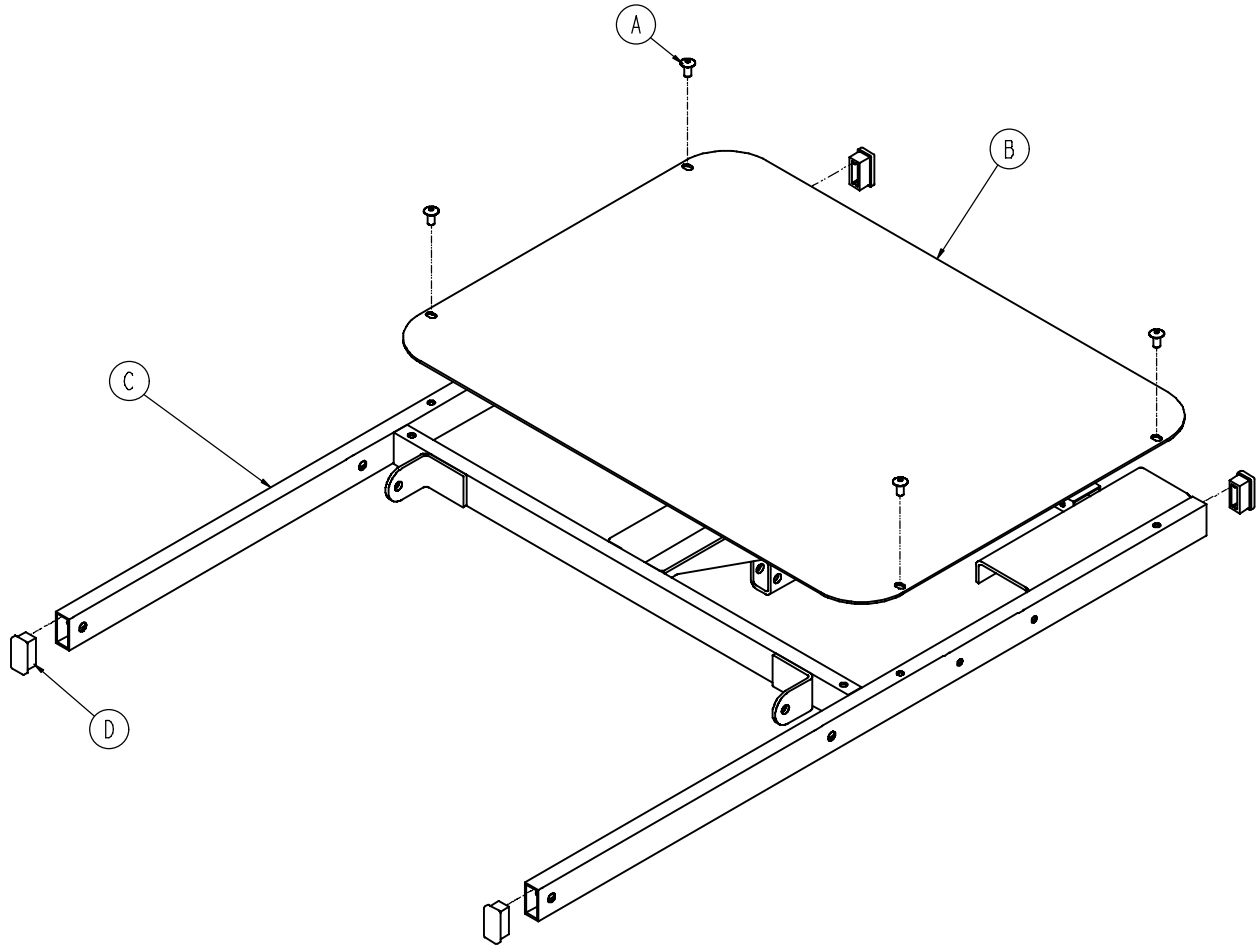

A	0025-079-000	4
B	0785-013-319	1
C	0785-013-325	1
D	1010-032-066	4


0785-014-602

Assembly part number 0785-014-602 (reference only)

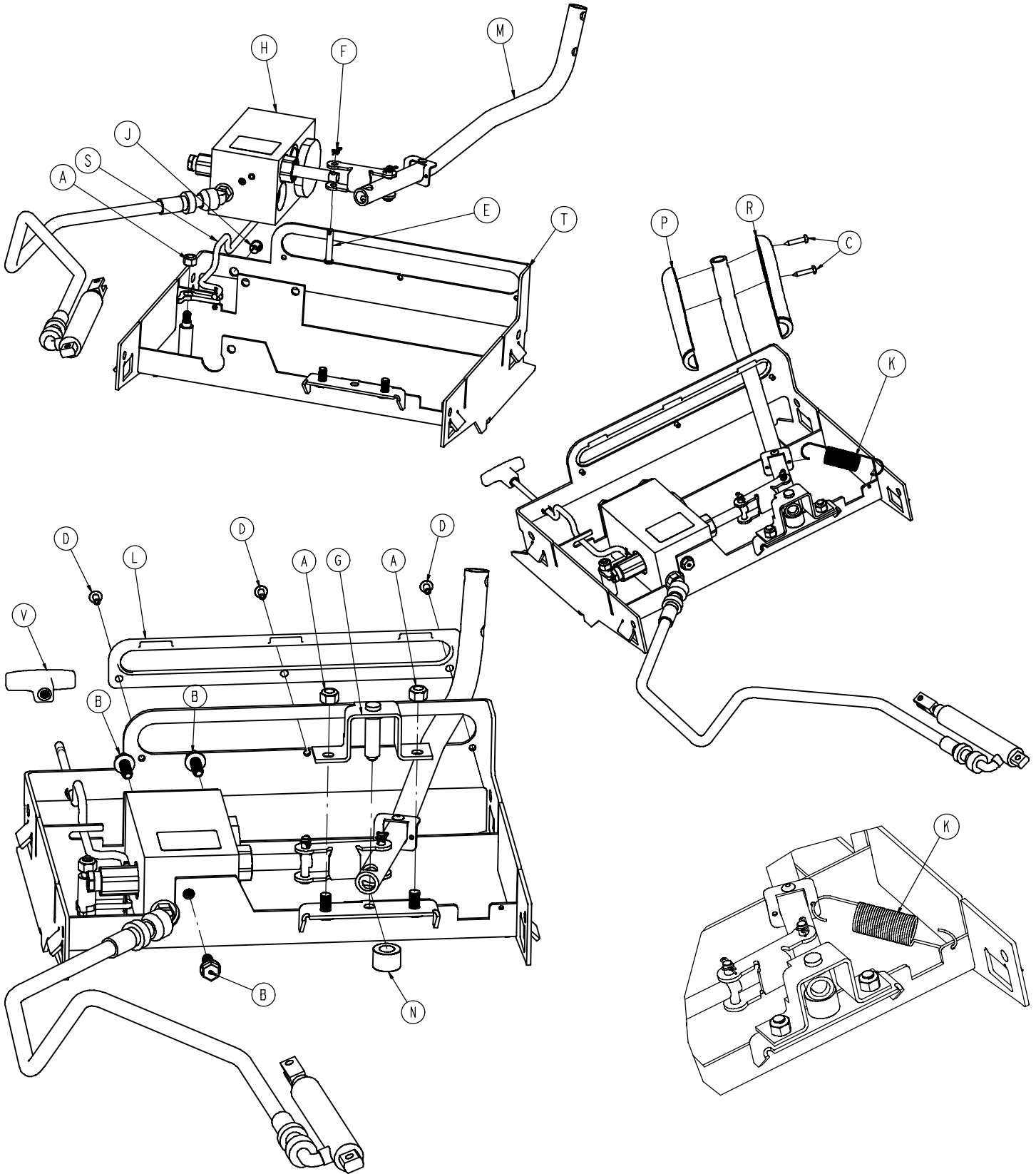

0785-014-602


A	0016-036-000	3
B	0023-288-000	3
C	0023-310-000	2
D	0025-079-000	3
E	8805-932-000	1
F	0027-020-000	1
G	0785-014-030	1
H	0785-014-043	1
J	0785-014-081	1
K	0785-014-107	1
L	0785-014-113	1
M	0785-014-116	1
N	0785-014-133	1
P	0785-014-156	1
R	0785-014-157	1
S	0785-014-603	1
T	0785-014-625	1
V	0785-114-027	1

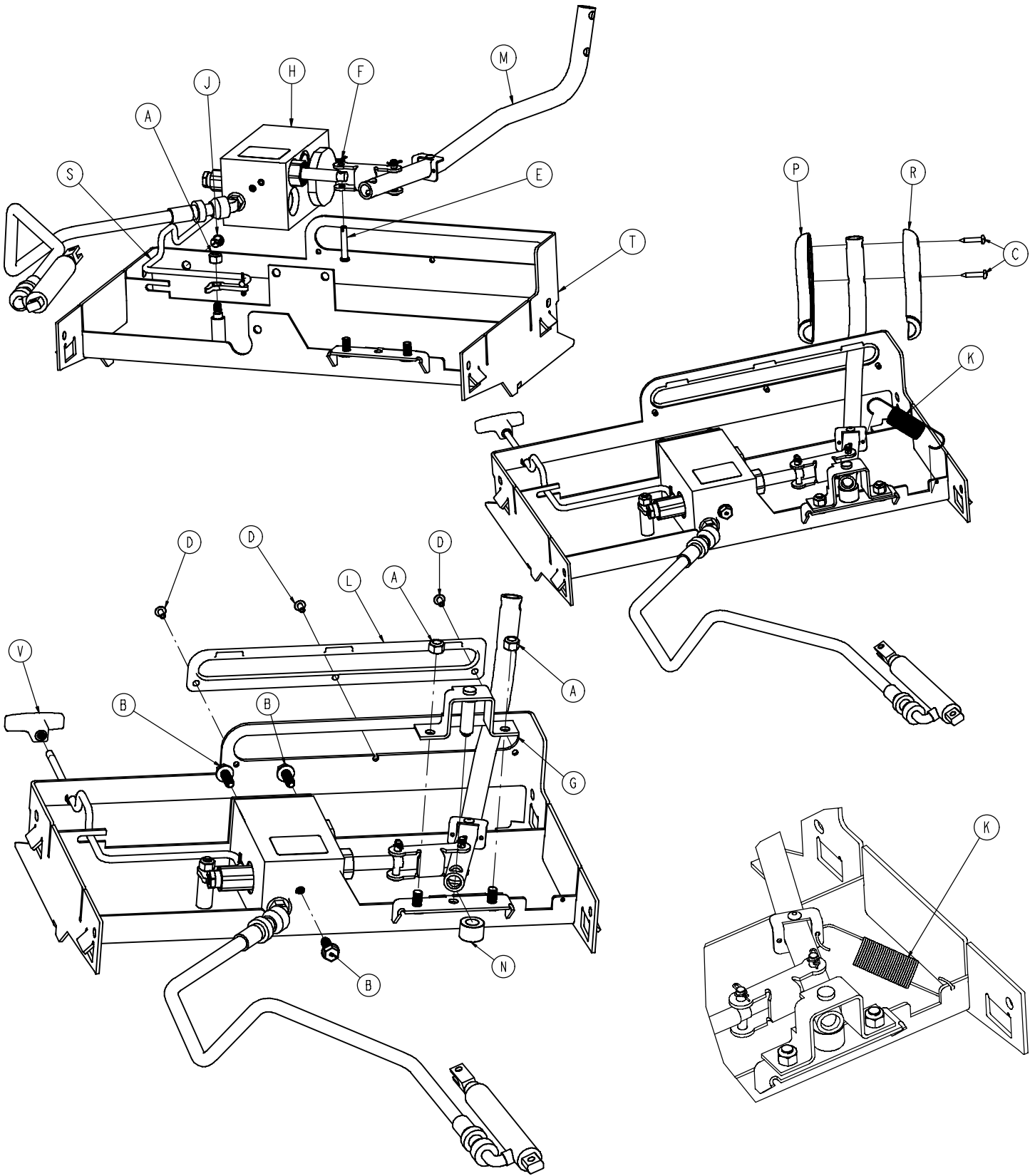

0785-014-302


A	0016-036-000	3
B	0023-288-000	3
C	0023-310-000	2
D	0025-079-000	3
E	8805-932-000	1
F	0027-020-000	1
G	0785-014-030	1
H	(page 69)	1
J	0785-014-081	1
K	0785-014-107	1
L	0785-014-113	1
M	(page 71)	1
N	0785-014-133	1
P	0785-014-156	1
R	0785-014-157	1
S	(page 72)	1
T	0785-014-325	1
V	0785-114-027	1

0785-700-001

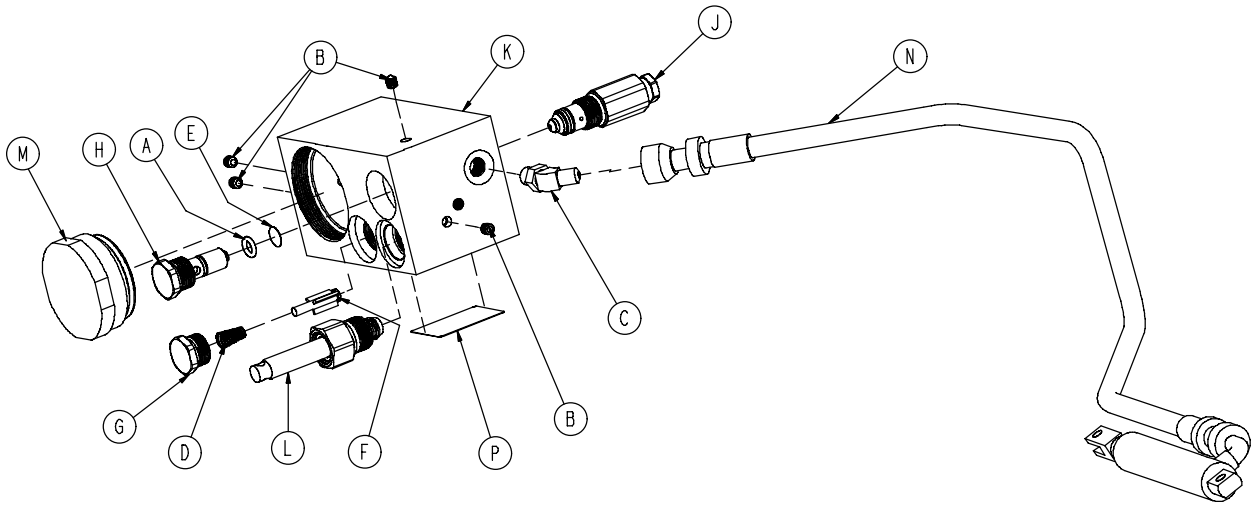


A	0046-004-000	1
B	0048-021-000	4
C	0048-175-000	1
D	0390-002-134	1
E	0715-001-321	1
F	0715-001-341	1
G	0753-002-047	1
H	0753-002-052	1
J	0753-002-065	1
K	0785-014-036	1
L	0785-014-051	1
M	0785-014-073	1
N	0785-014-078	1
P	0785-014-114	1

0785-014-078

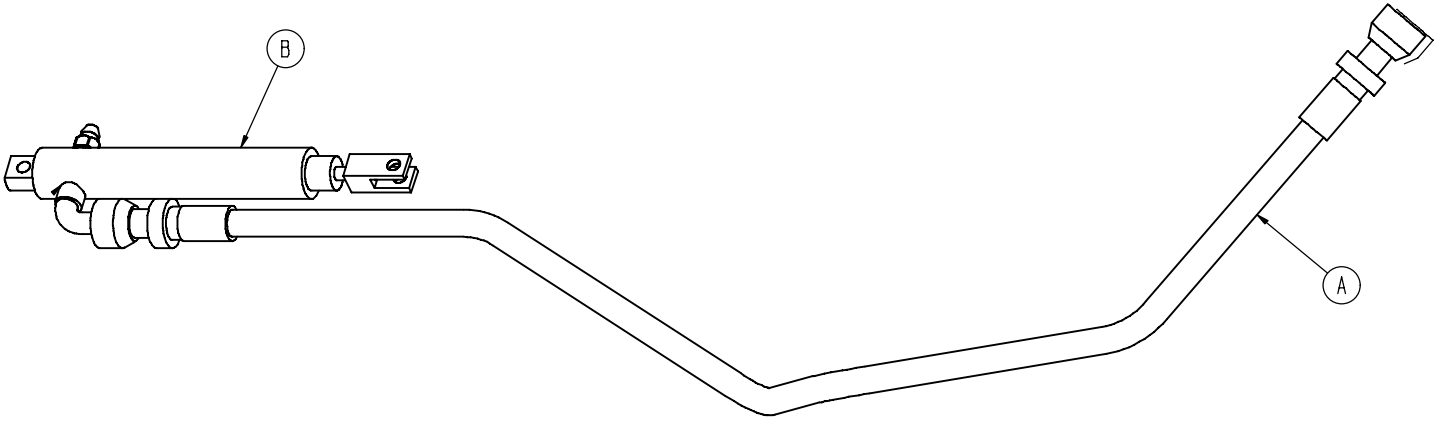


A	0785-014-071	1
B	0785-014-077	1


0785-014-116

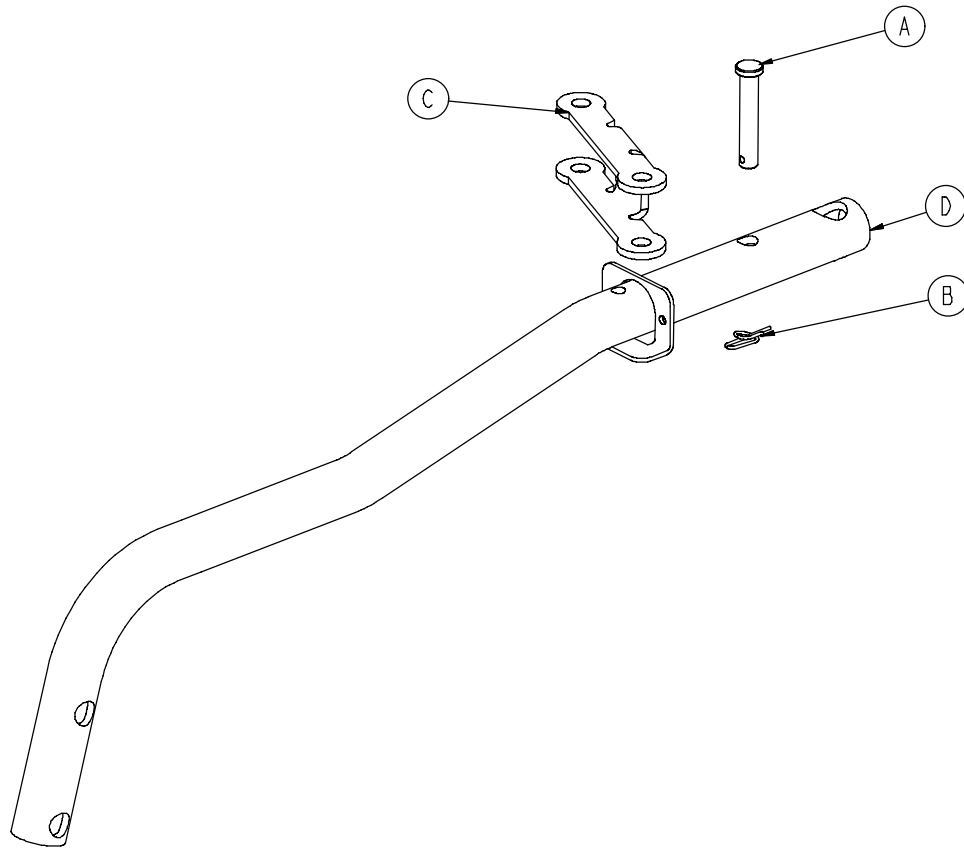

A	8805-932-000	1
B	0027-020-000	1
C	0785-014-024	1
D	0785-014-111	1

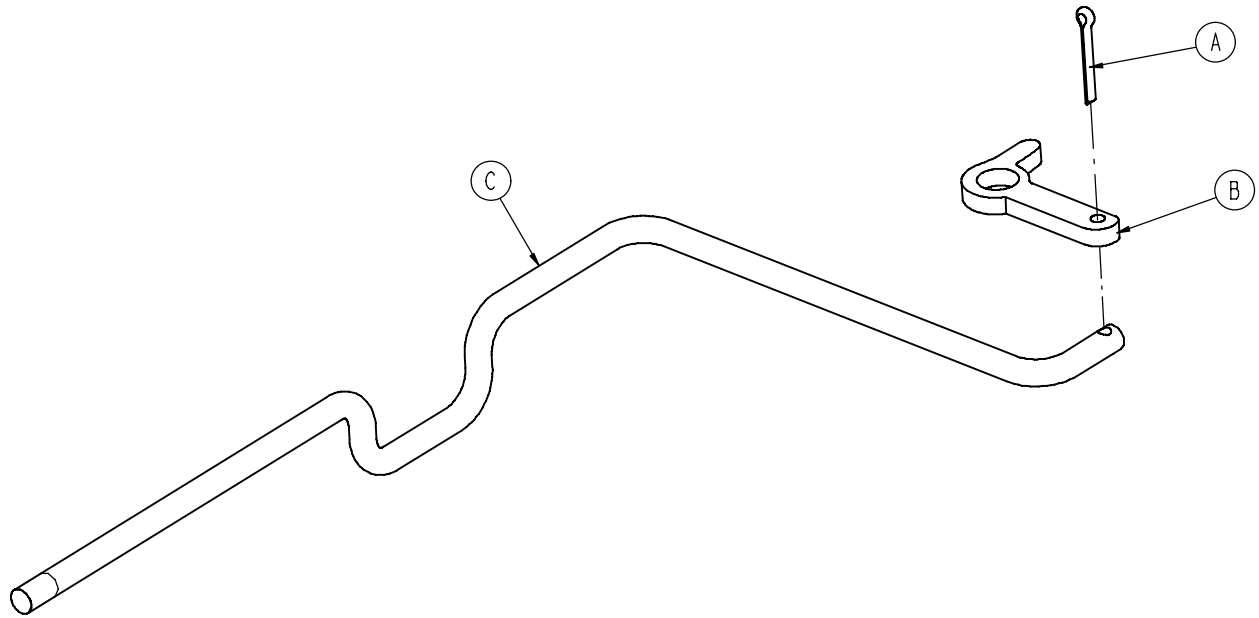


A	0027-015-000	1
B	0785-014-038	1
C	0785-014-337	1

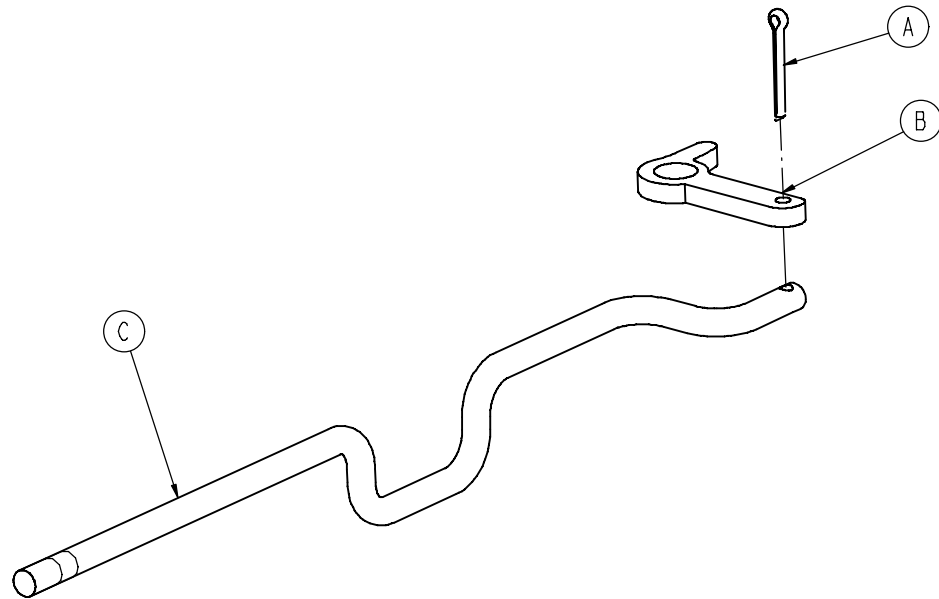


A	0027-015-000	1
B	0785-014-038	1
C	0785-014-637	1

0785-014-610

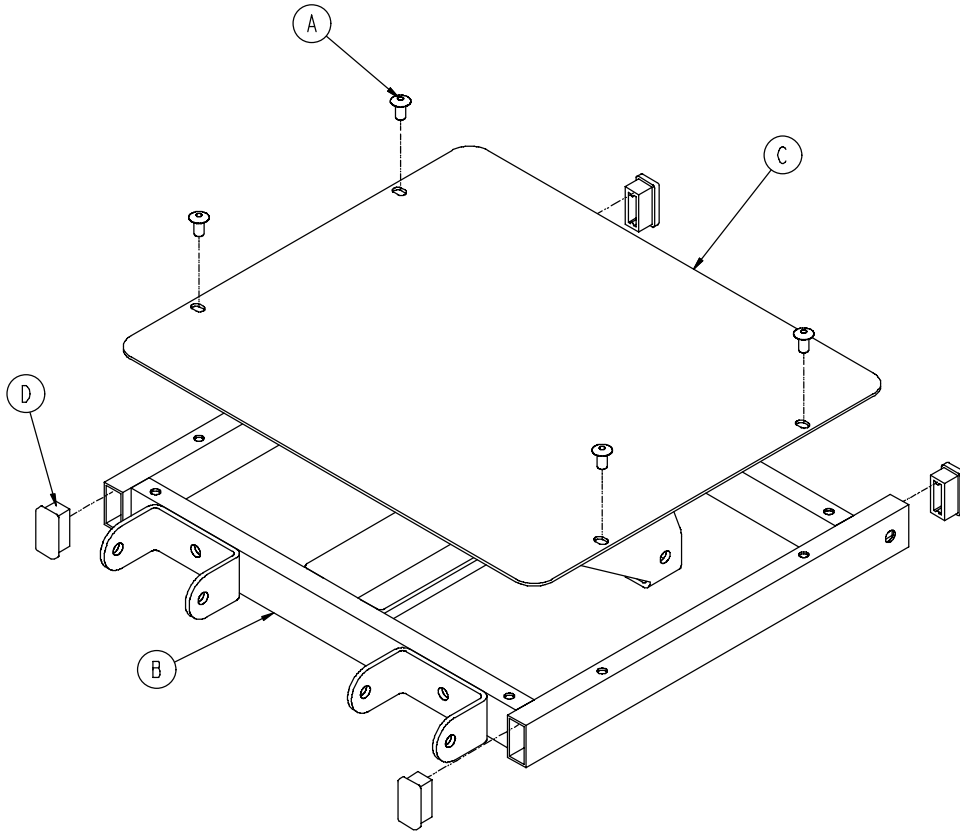


A	0025-079-000	4
B	0785-014-615	1
C	0785-014-619	1
D	1010-032-066	4

0785-014-310

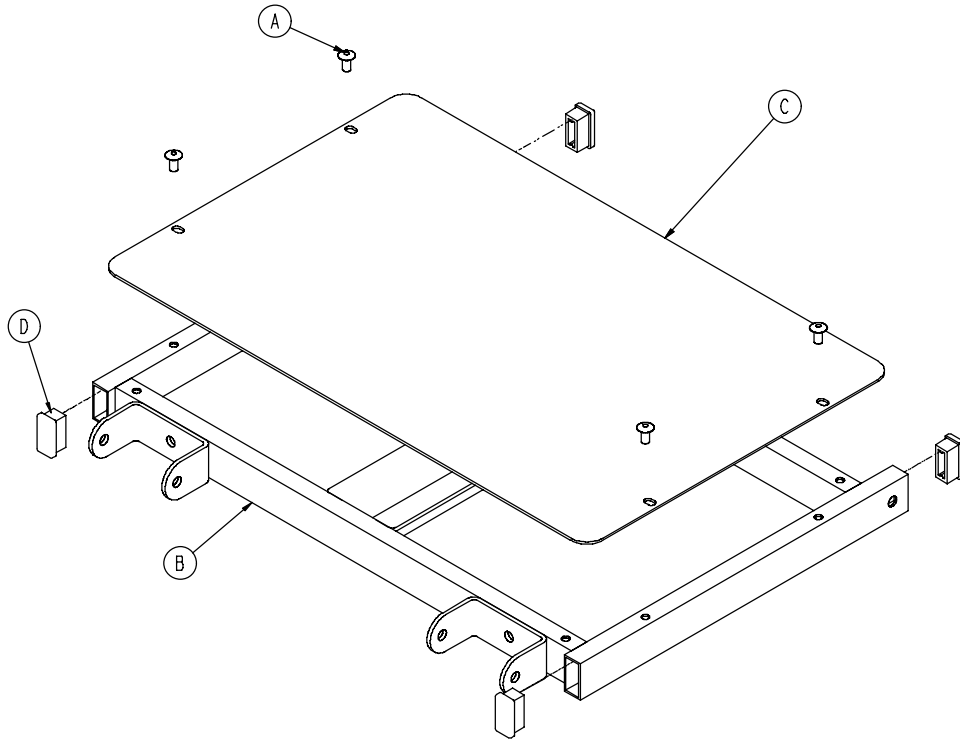


A	0025-079-000	4
B	0785-014-315	1
C	0785-014-319	1
D	1010-032-066	4

0785-015-601

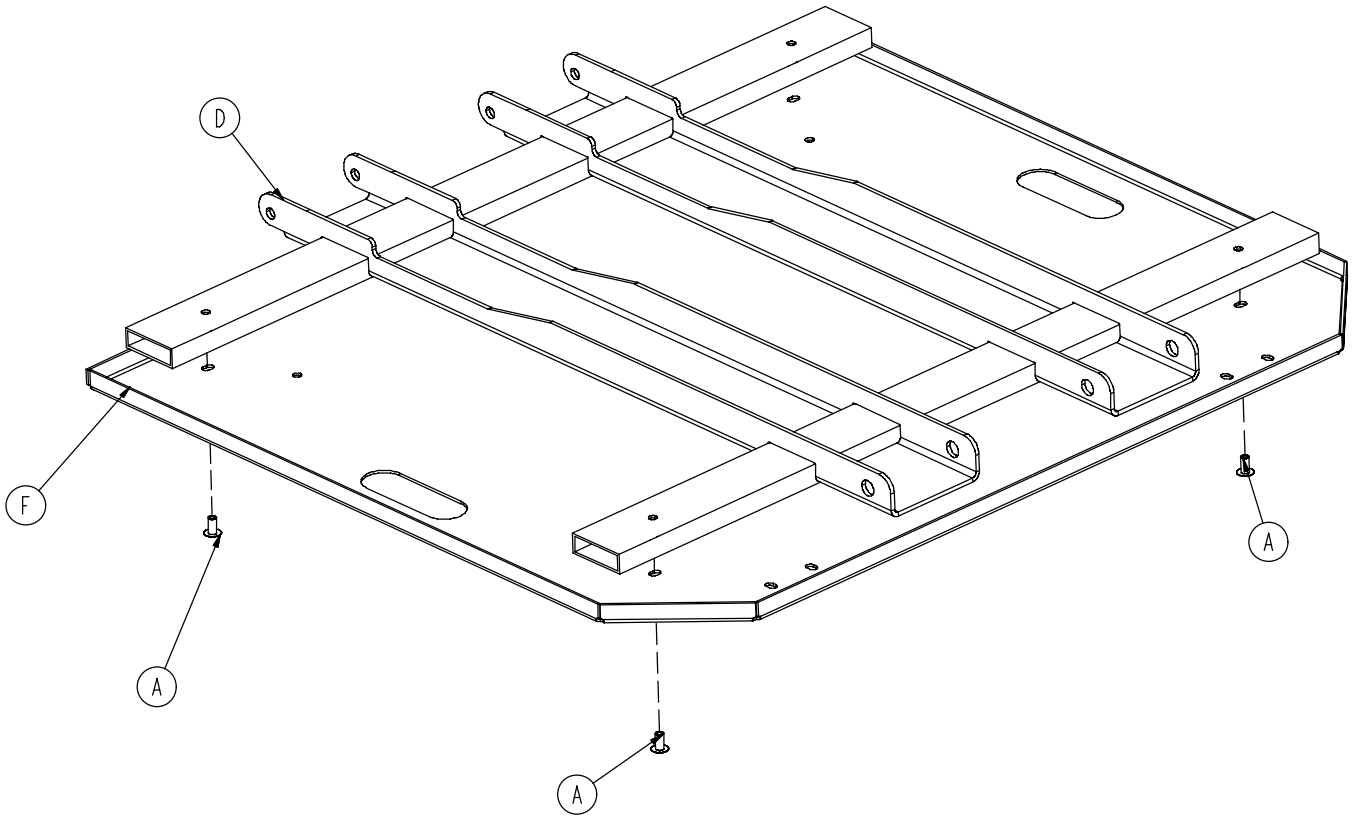


A	0025-079-000	4
D	0785-015-605	1
F	0785-015-623	1


0785-015-301

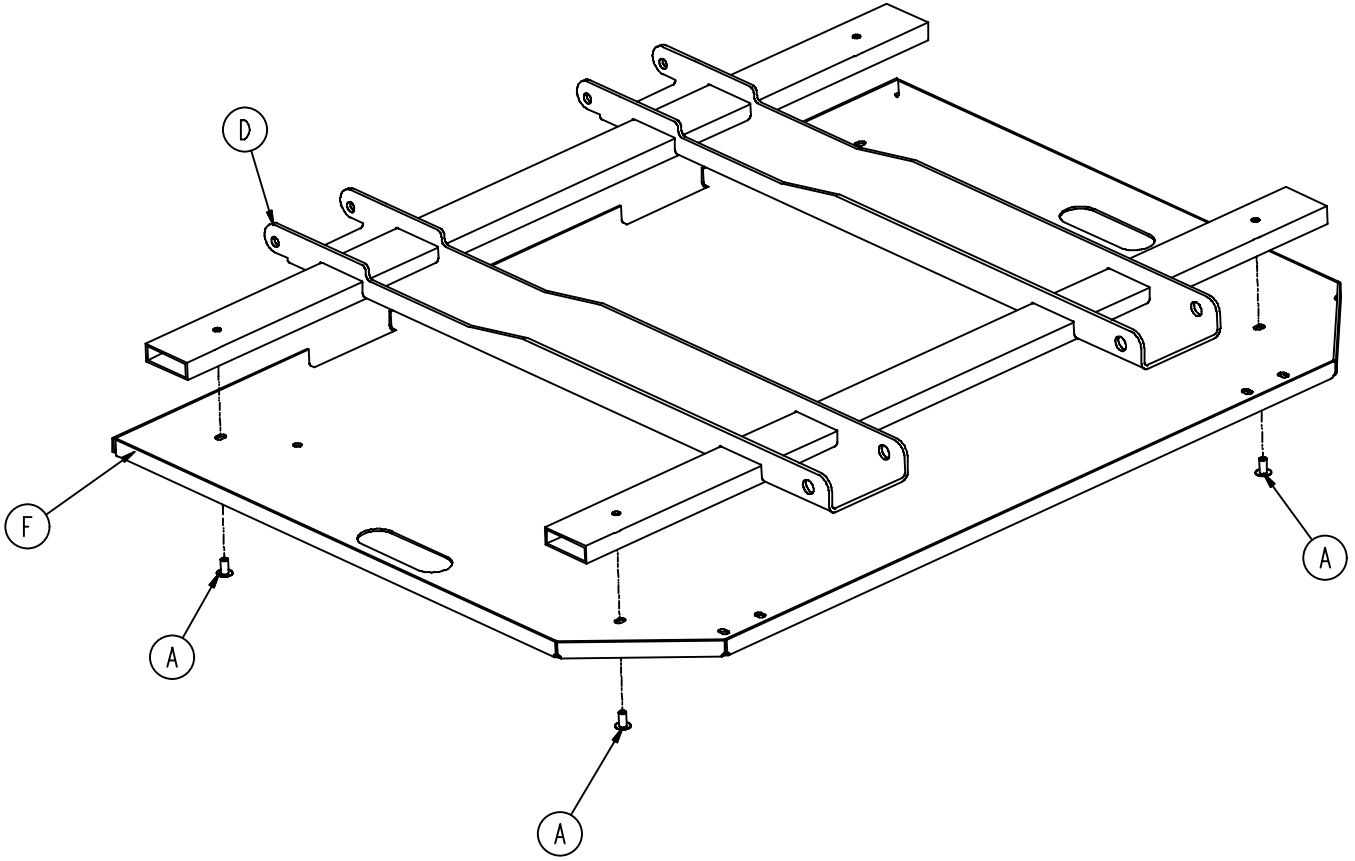


A	0025-079-000	4
D	0785-015-305	1
F	0785-015-323	1

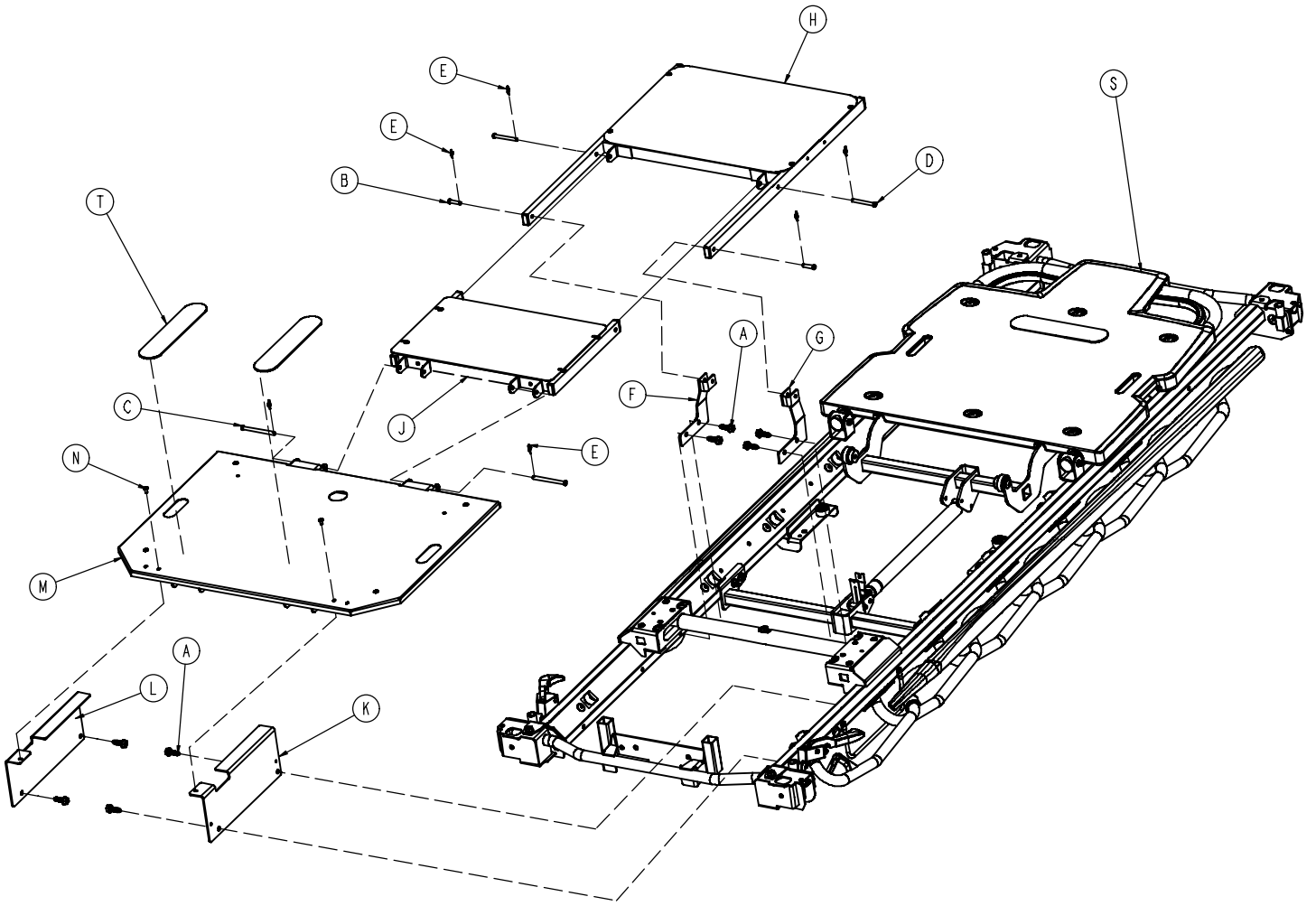


0785-010-604

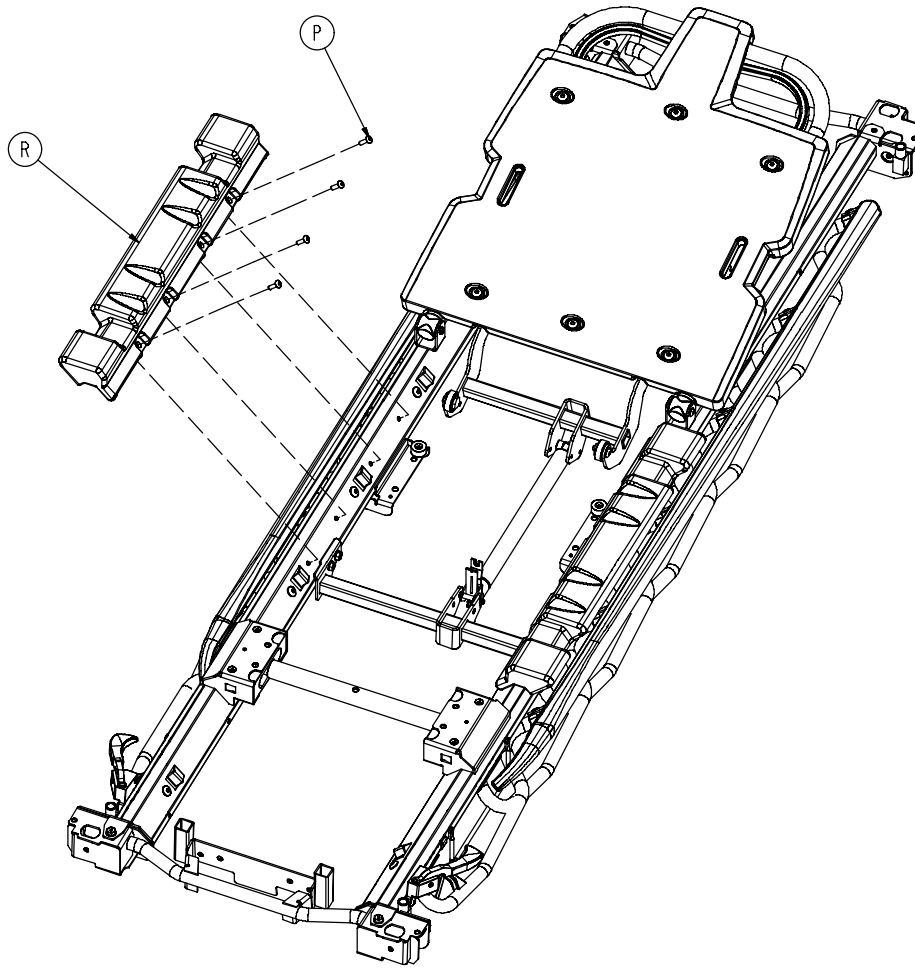


A	0023-288-000	8
B	0026-316-000	2
C	0026-364-000	2
D	0026-334-000	2
E	0027-020-000	6
F	0785-013-034	1
G	0785-013-039	1
H	0785-013-610	1
J	0785-014-610	1
K	0785-015-016	1
L	0785-015-018	1
M	0785-015-601	1
N	0025-079-000	2
P	0025-147-000	8
R	0785-010-049	2
S	0785-012-301	1
T	0785-034-005	2

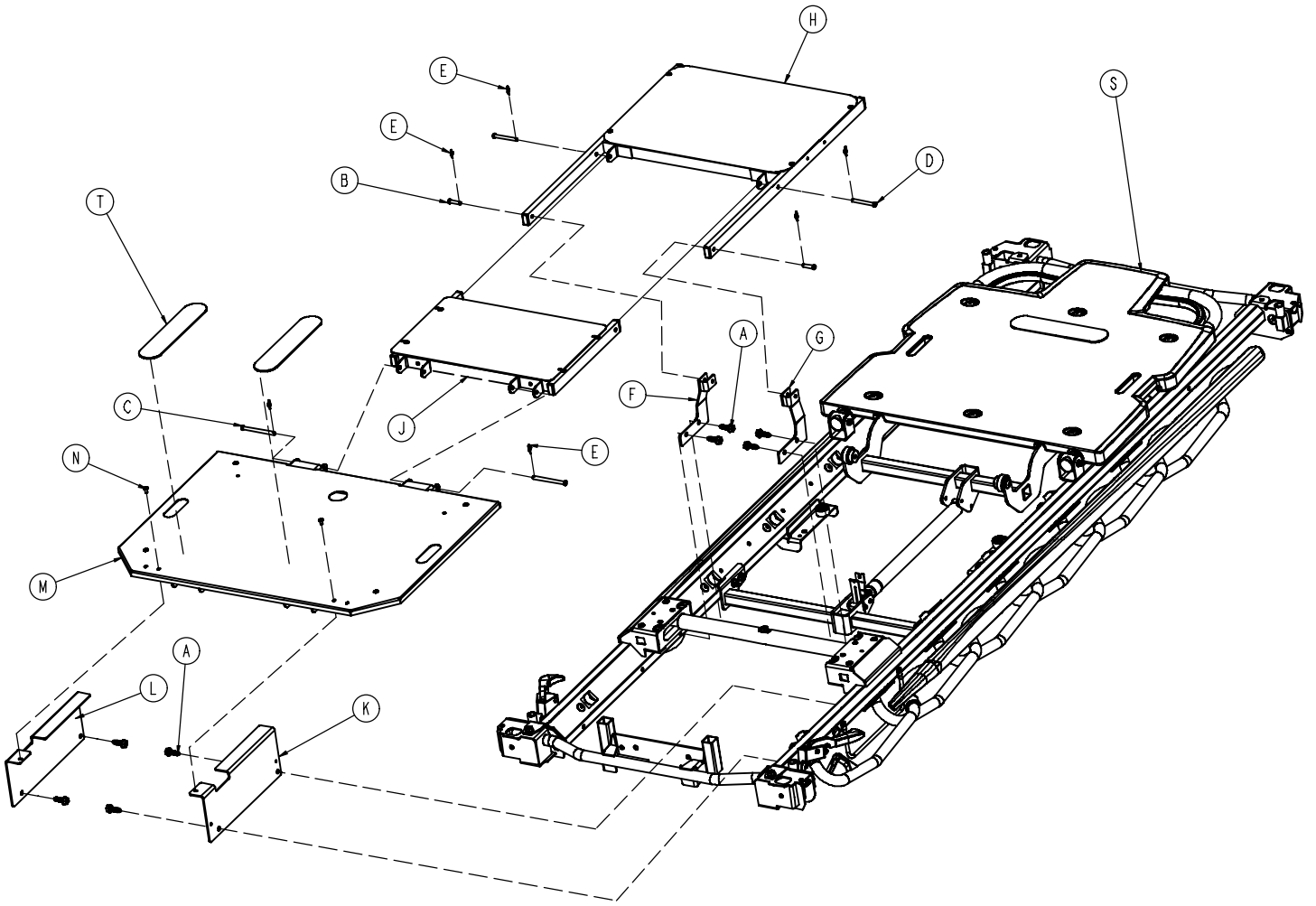

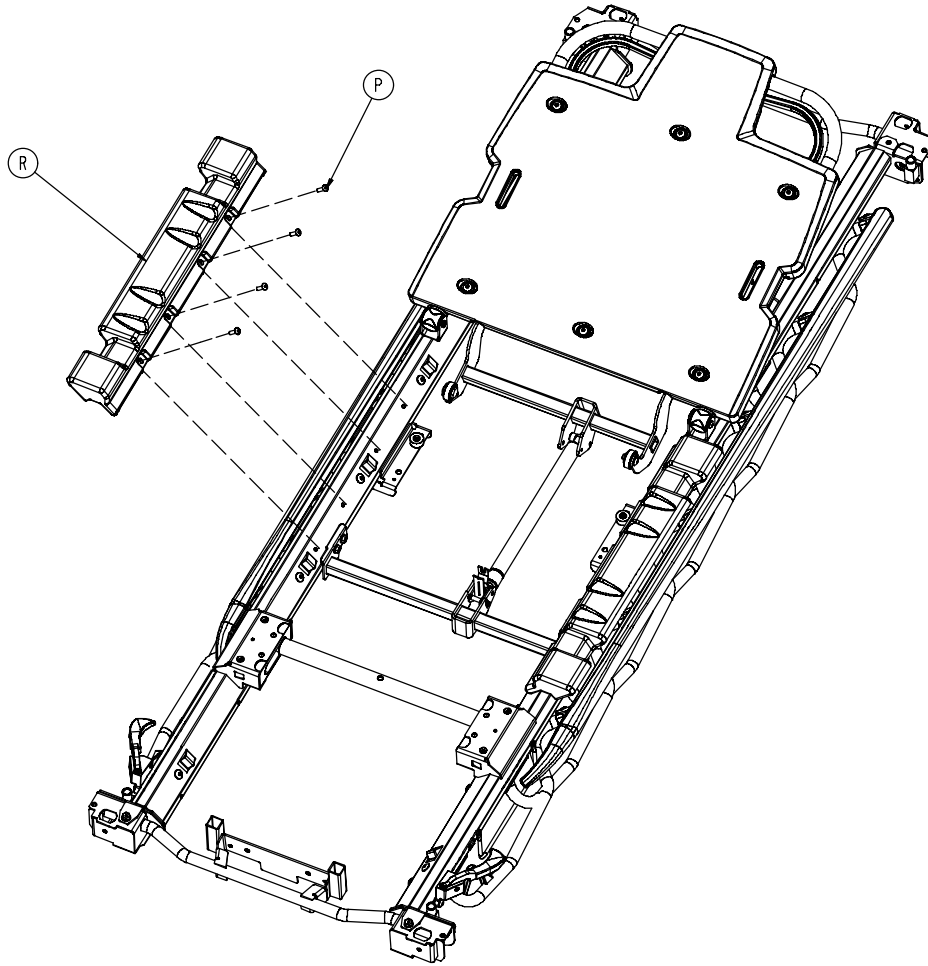


A	0023-288-000	8
B	0026-316-000	2
C	0026-364-000	2
D	0026-334-000	2
E	0027-020-000	6
F	0785-013-034	1
G	0785-013-039	1
H	(page 64)	1
J	(page 74)	1
K	0785-015-016	1
L	0785-015-018	1
M	(page 76)	1
N	0025-079-000	2
P	0025-147-000	8
R	0785-010-049	2
S	(page 81)	1
T	0785-034-005	2

0785-012-301

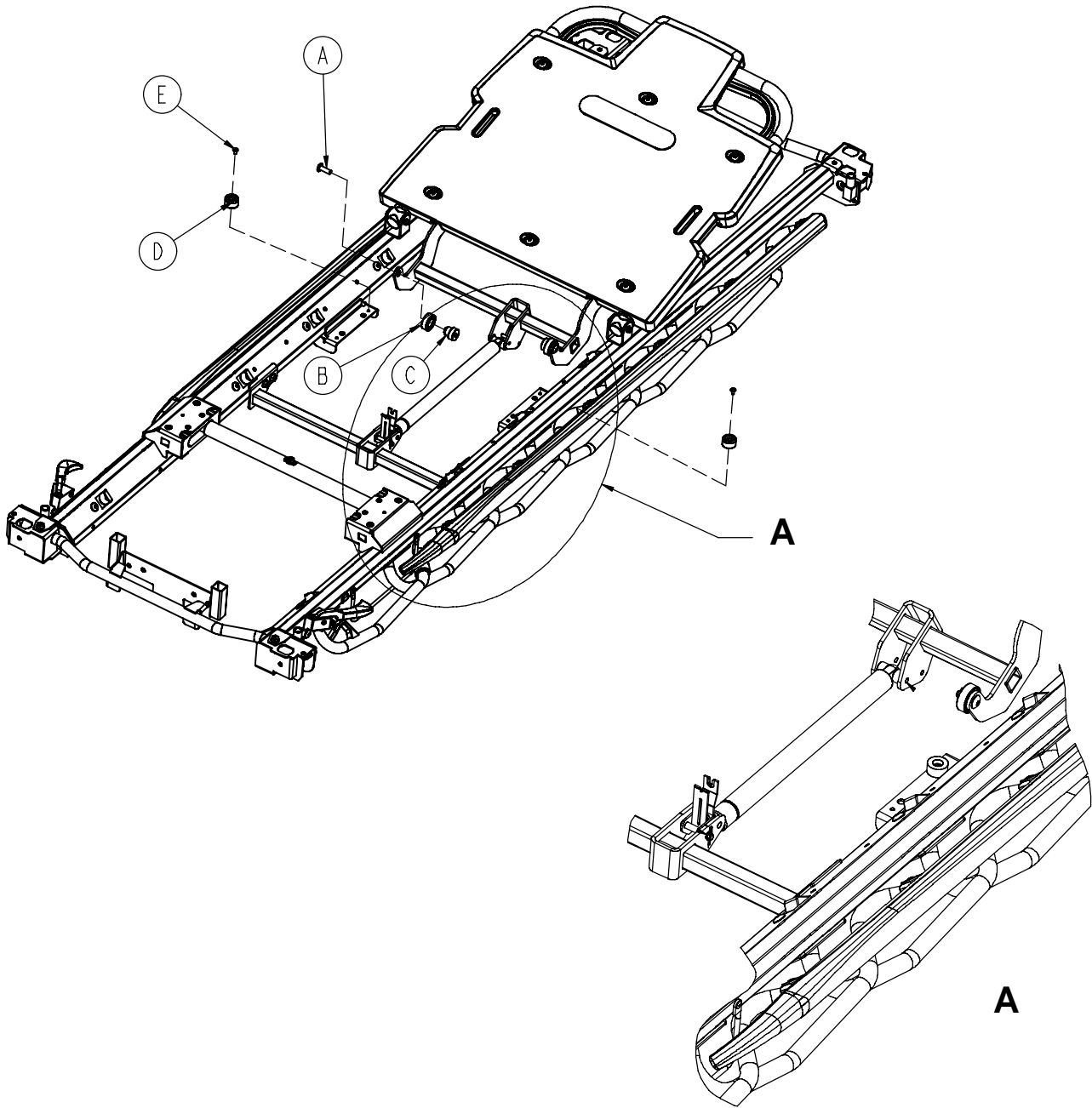


A	0007-074-000	2
B	0785-012-007	2
C	0785-012-101	2
D	0056-023-000	2
E	0025-079-000	2

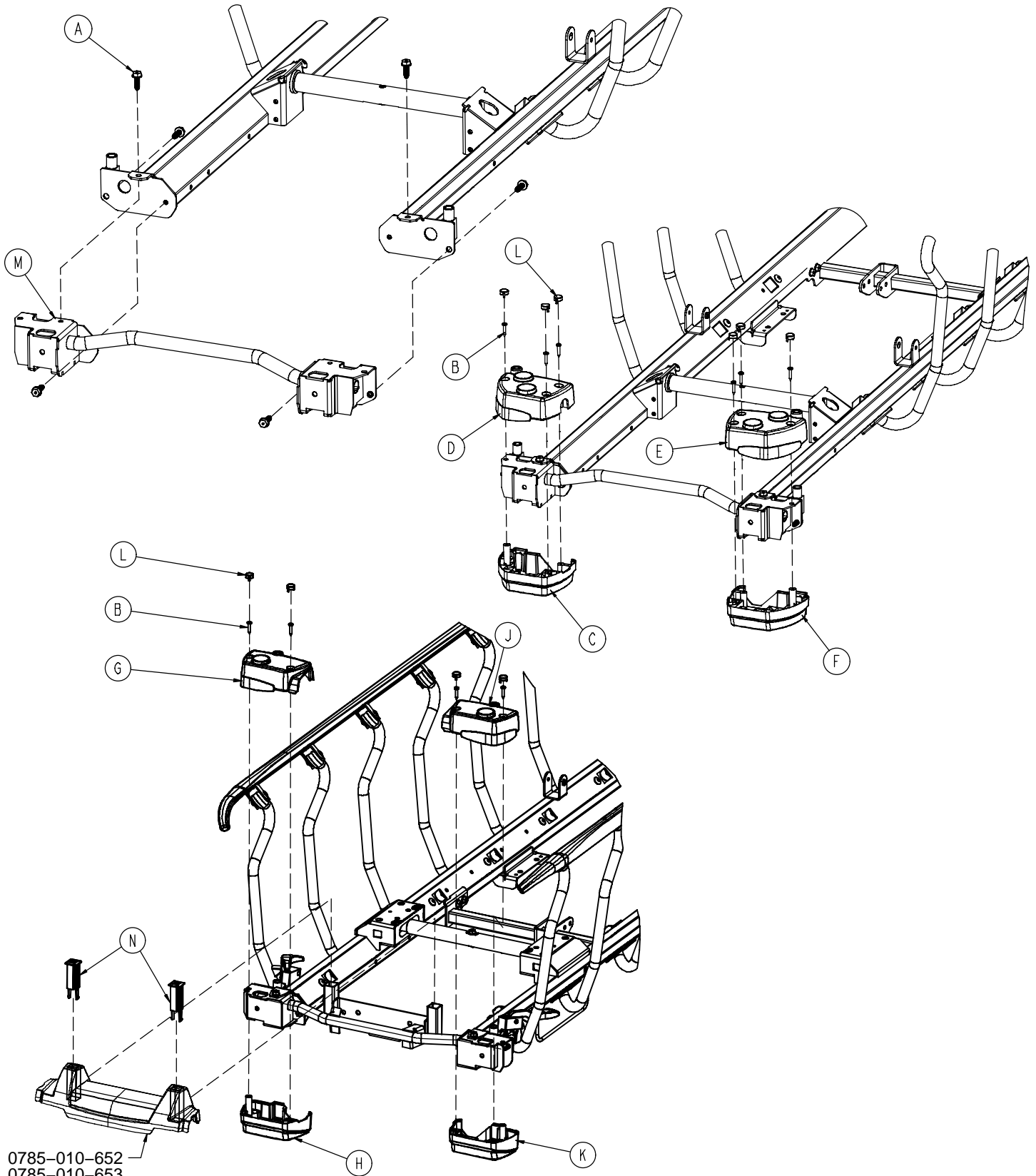

0785-010-652
0785-010-653

0785-010-304

A	0023-288-000	6
B	0023-310-000	10
C	0785-010-050	1
D	0785-010-051	1
E	0785-010-052	1
F	0785-010-053	1
G	0785-010-058	1
H	0785-010-059	1
J	0785-010-062	1
K	0785-010-063	1
L	0785-010-140	10
M	0785-010-615	1
N	0785-010-122	2

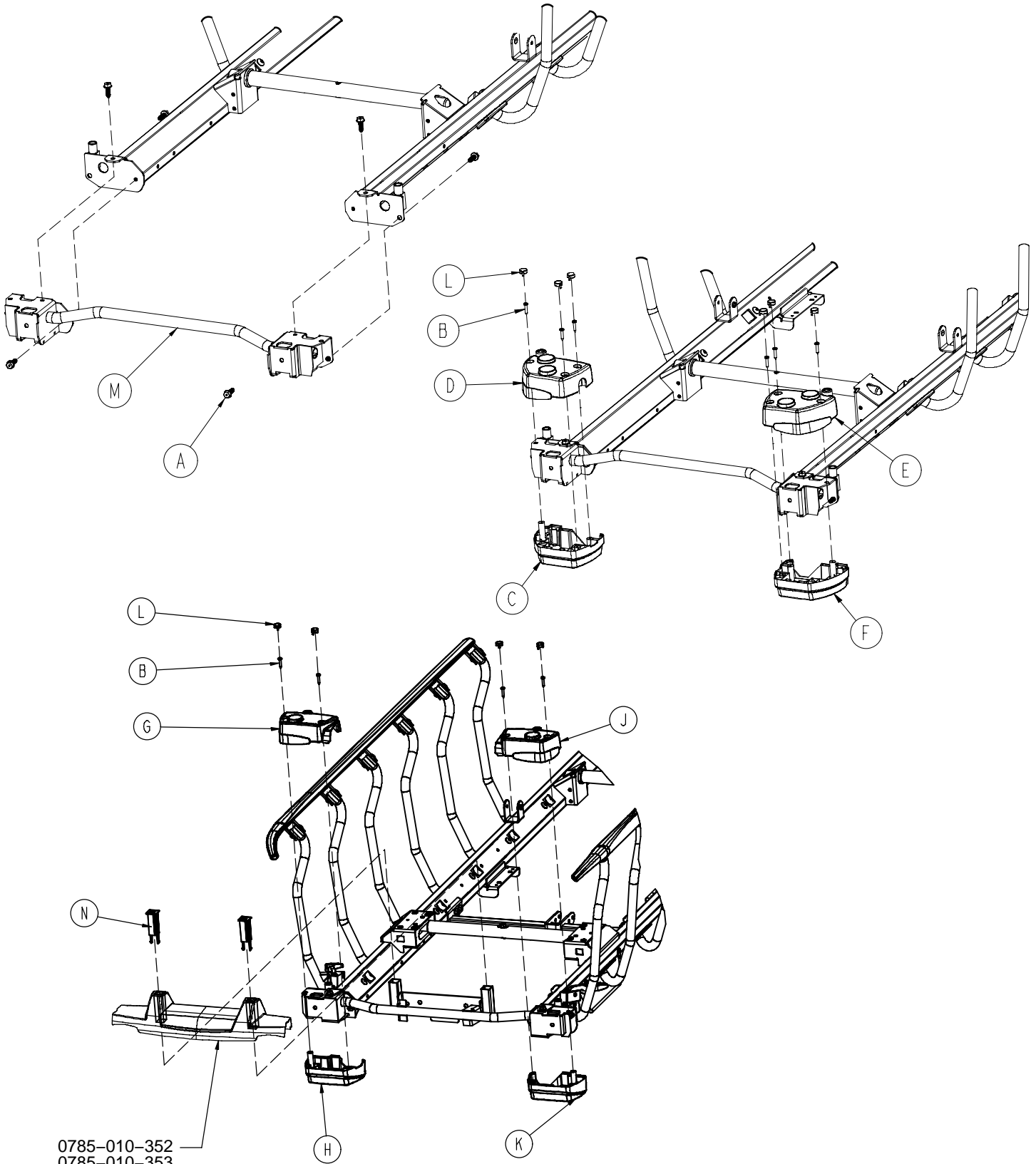


0785-010-352
0785-010-353

0785-010-150


A	0023-288-000	6
B	0023-310-000	10
C	0785-010-050	1
D	0785-010-051	1
E	0785-010-052	1
F	0785-010-053	1
G	0785-010-058	1
H	0785-010-059	1
J	0785-010-062	1
K	0785-010-063	1
L	0785-010-140	10
M	0785-010-315	1
N	0785-010-122	2

0785-045-610

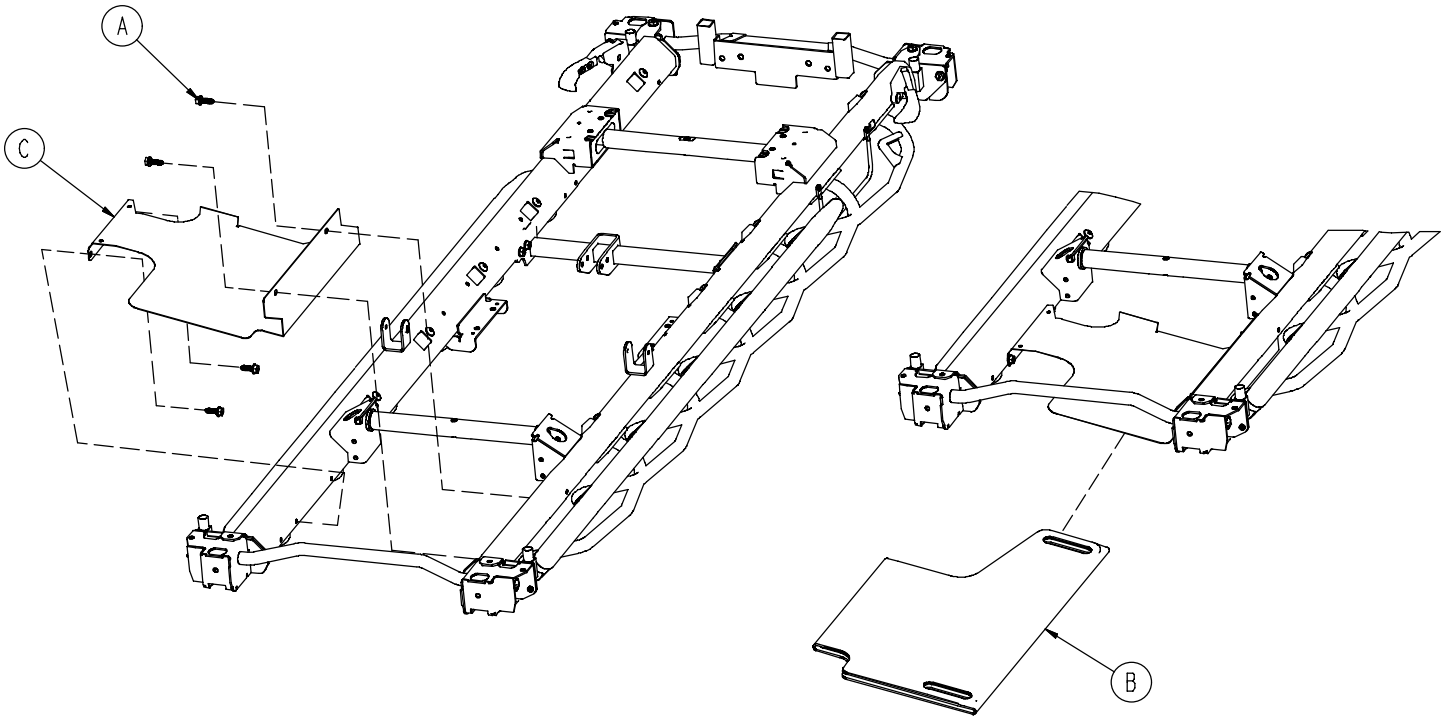


A	0023-288-000	4
B	0785-018-001	1
C	0785-045-611	1


0785-045-310

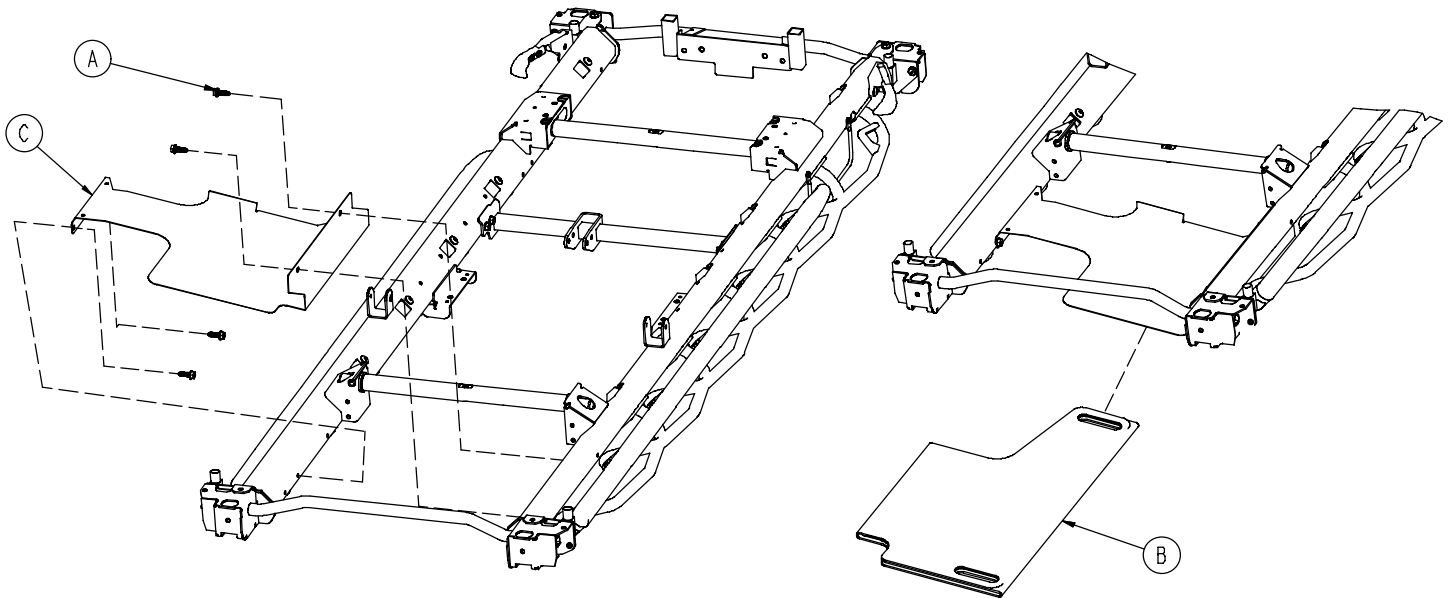

A	0023-288-000	4
B	0785-018-001	1
C	0785-045-311	1

0785-018-001

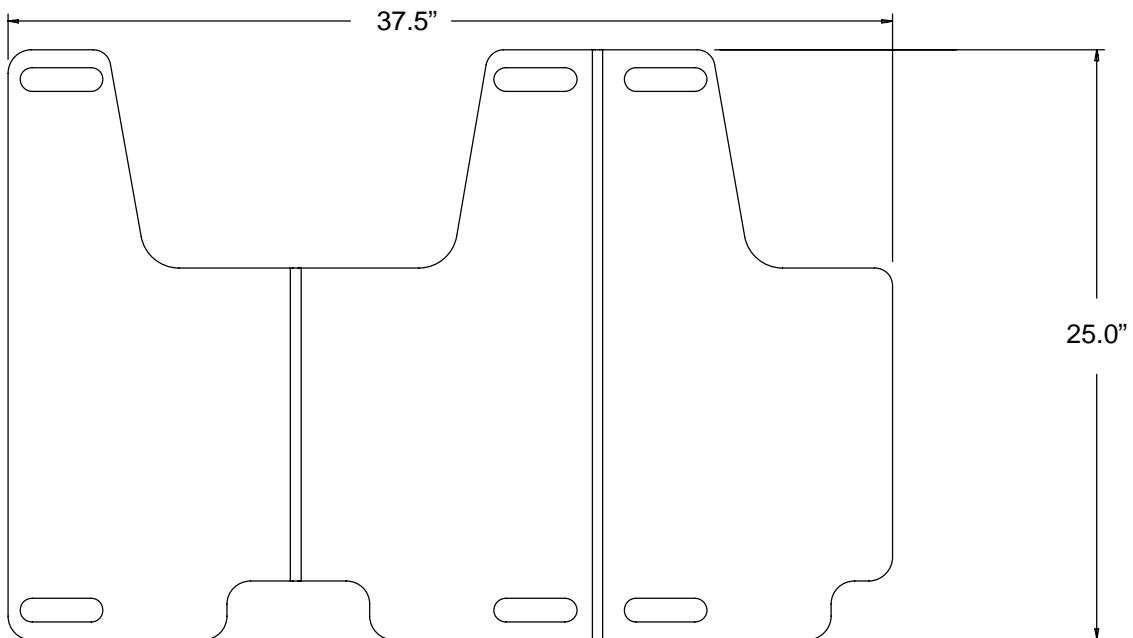


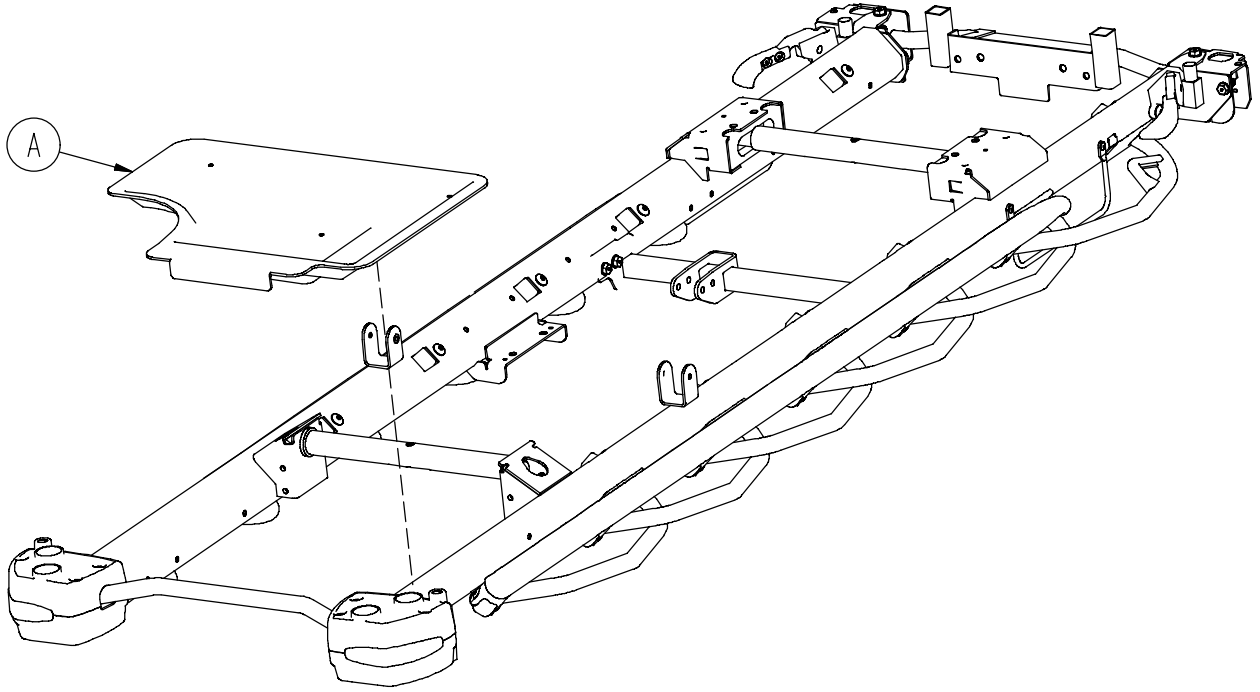

A

0785-045-601

1

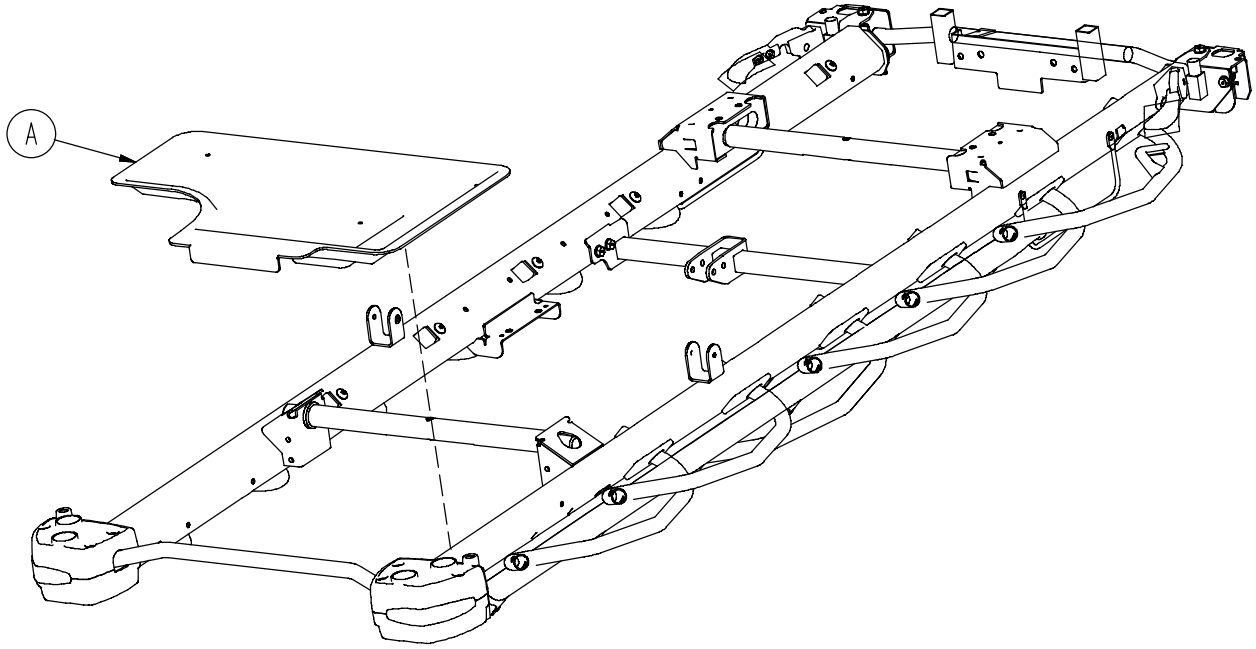


A

0785-045-301


1

0785-048-020

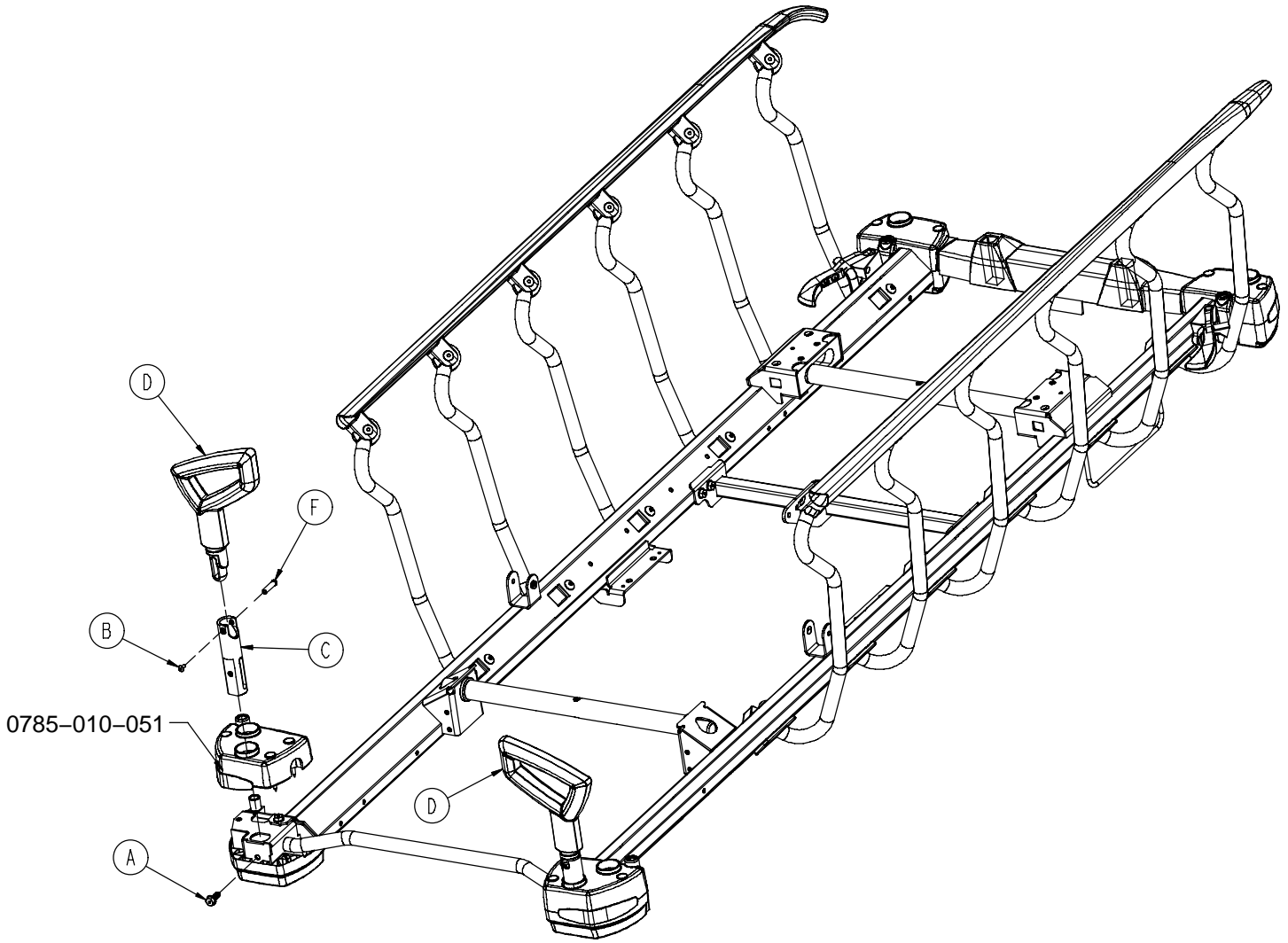

A	0023-288-000	2
B	0025-079-000	2
C	0785-010-020	2
D	0785-048-018	2
F	0785-048-025	2

0785-048-030

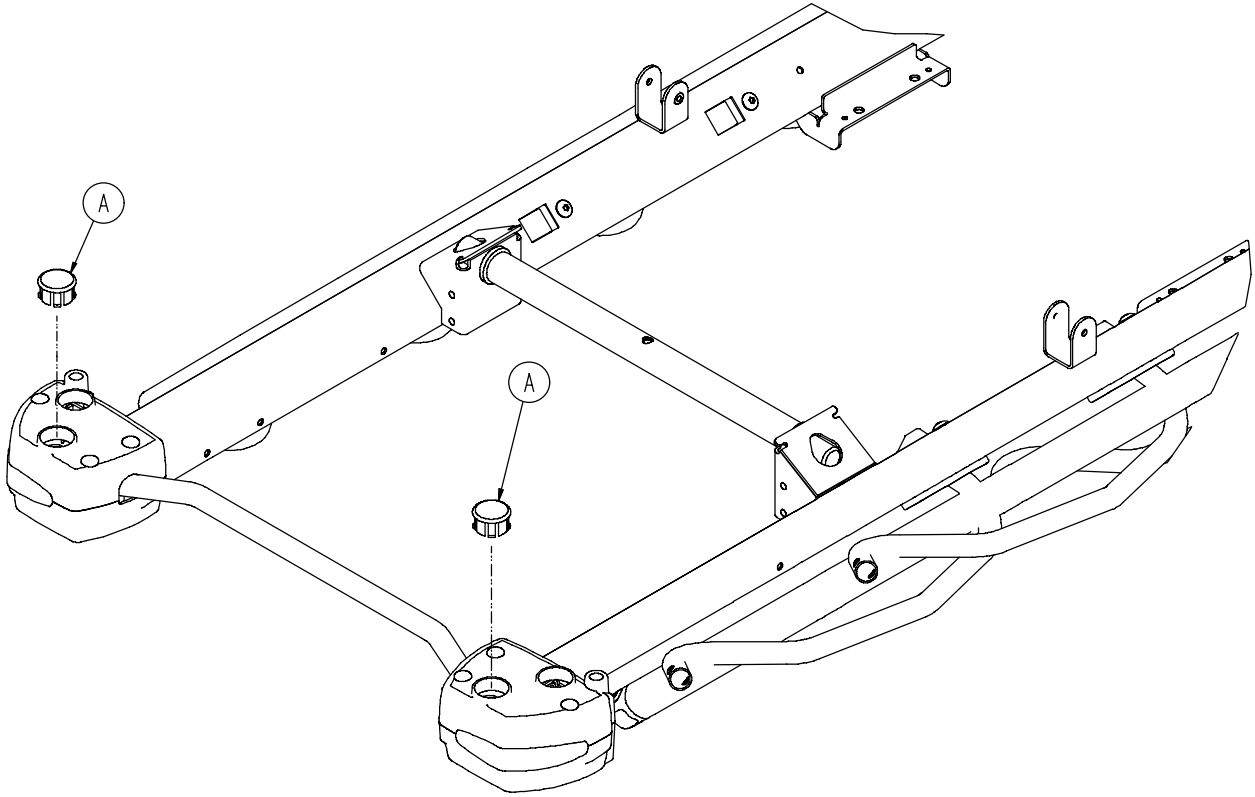


A


0785-010-131

2

0390-025-000

A	0024-023-000	1
B	0390-003-053	1
C	0393-003-043	1
D	0004-496-000	1

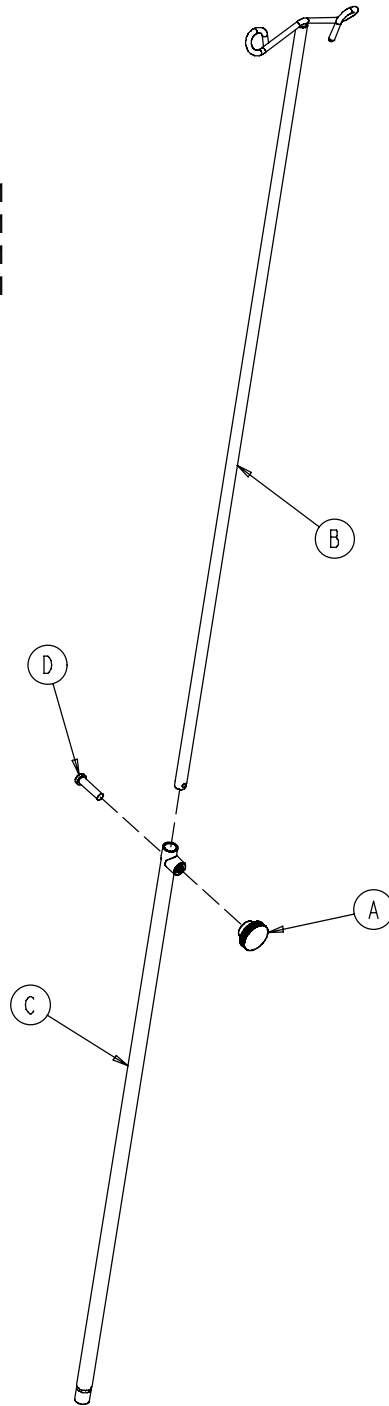

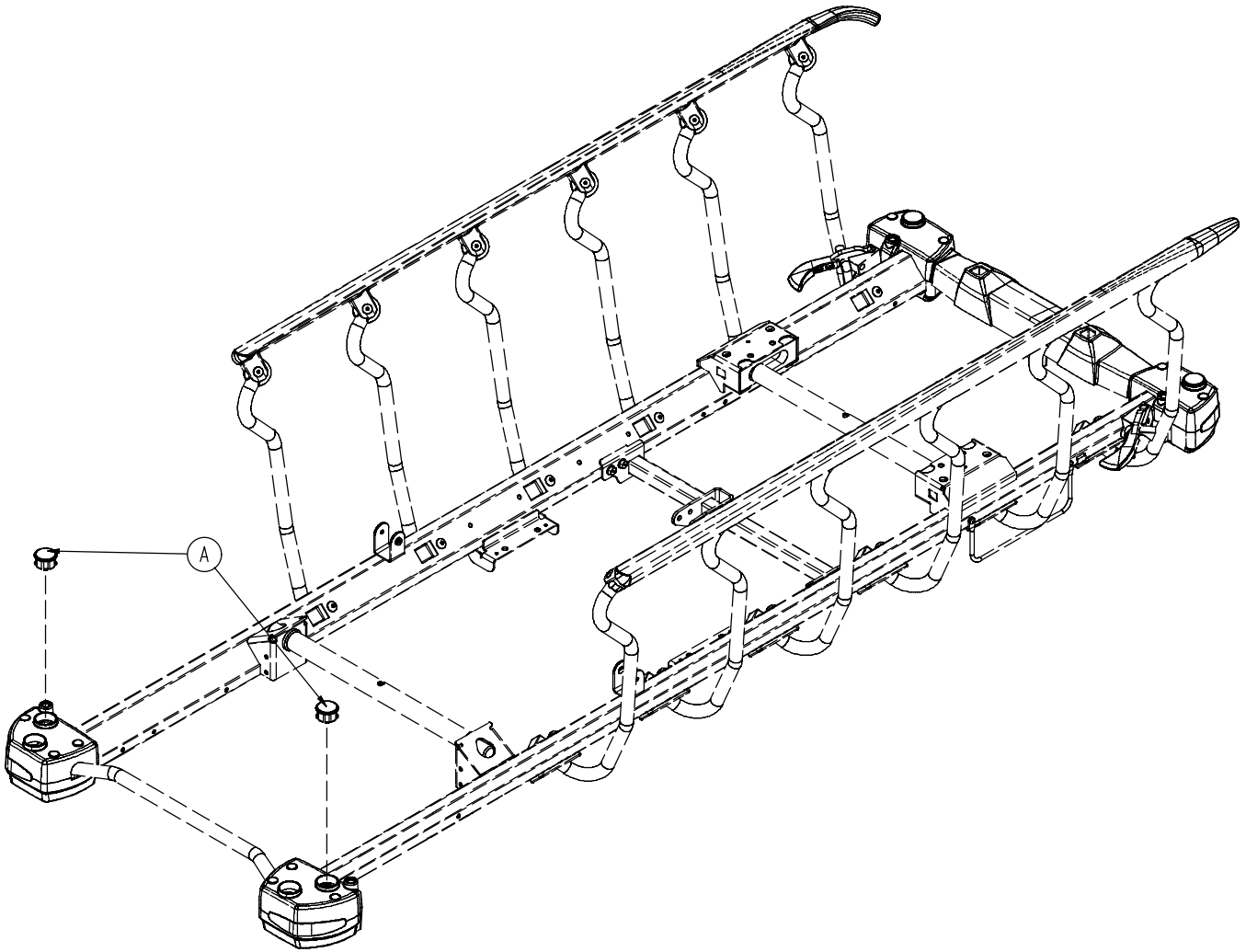


A

0785-010-131

2

0785-035-638/0785-035-641/0785-035-640/0785-035-643

0785-035-338/0785-035-341/0785-035-340/0785-035-343

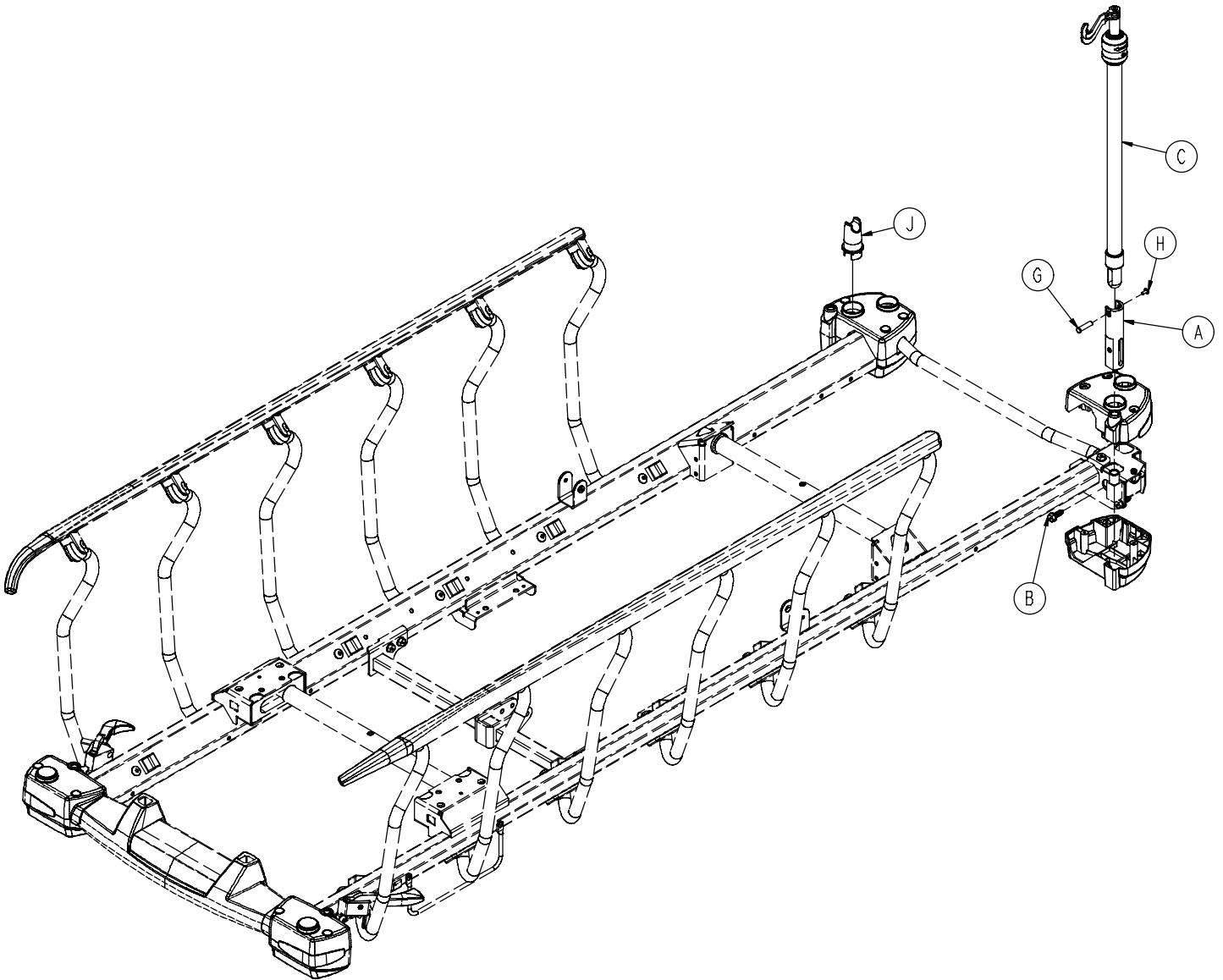


0785-035-638 / 0785-035-641 / 0785-035-640 / 0785-035-643

A	0785-010-020	1
B	0023-288-000	1
C	0785-035-101	1
G	0785-048-025	1
H	0025-079-000	1
J	0785-035-004	1

0785-035-338 / 0785-035-341 / 0785-035-340 / 0785-035-343


A	0785-010-020	1
B	0023-288-000	1
C	0785-035-401	1
G	0785-048-025	1
H	0025-079-000	1
J	0785-035-004	1

0785-035-101

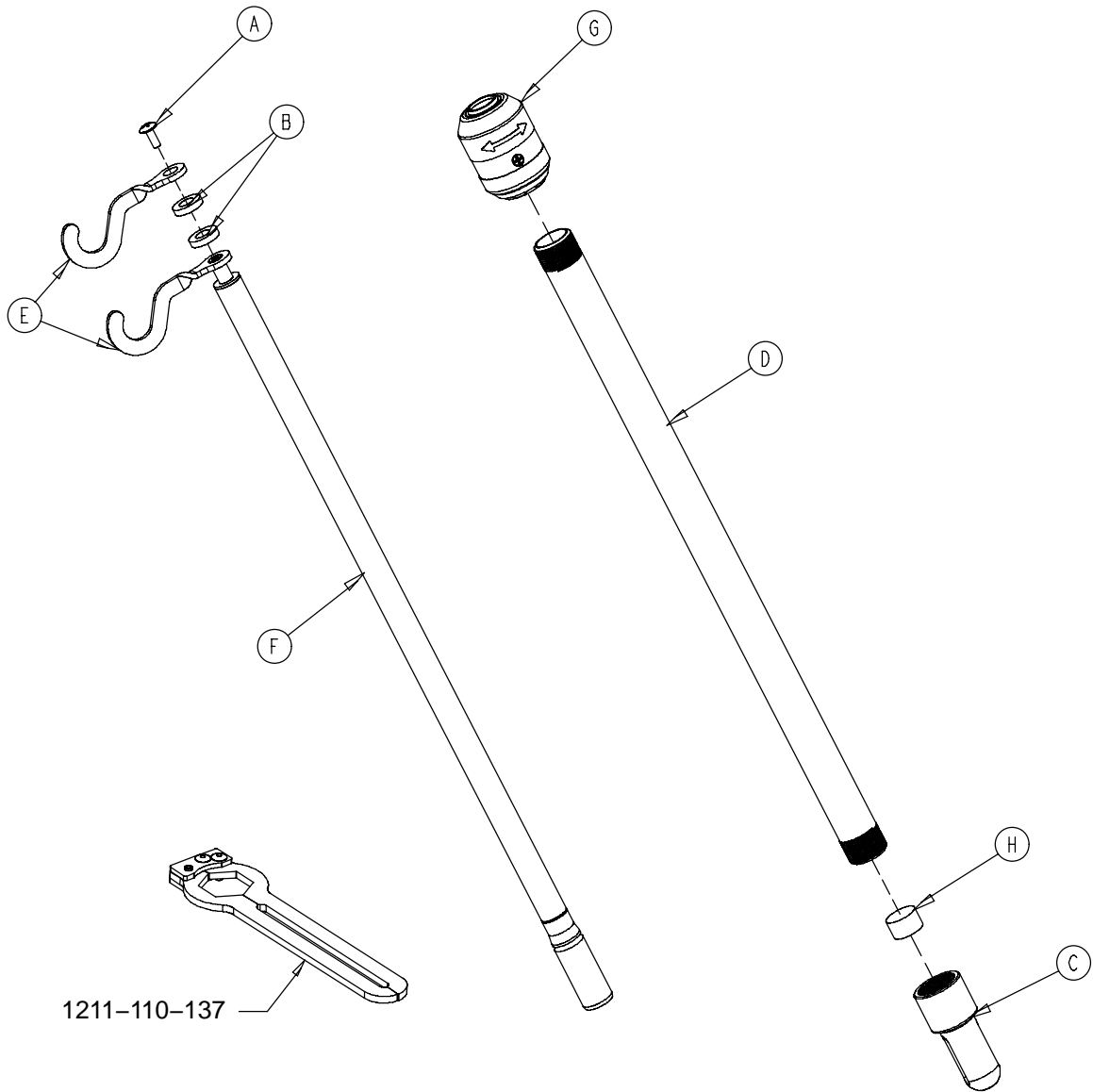

A	0007-040-000	1
B	0052-017-000	2
C	0785-035-003	1
D	0785-035-027	1
E	1010-059-016	2
F	0785-035-102	1
G	0785-035-103	1
H	1001-259-013	1


0785-035-401

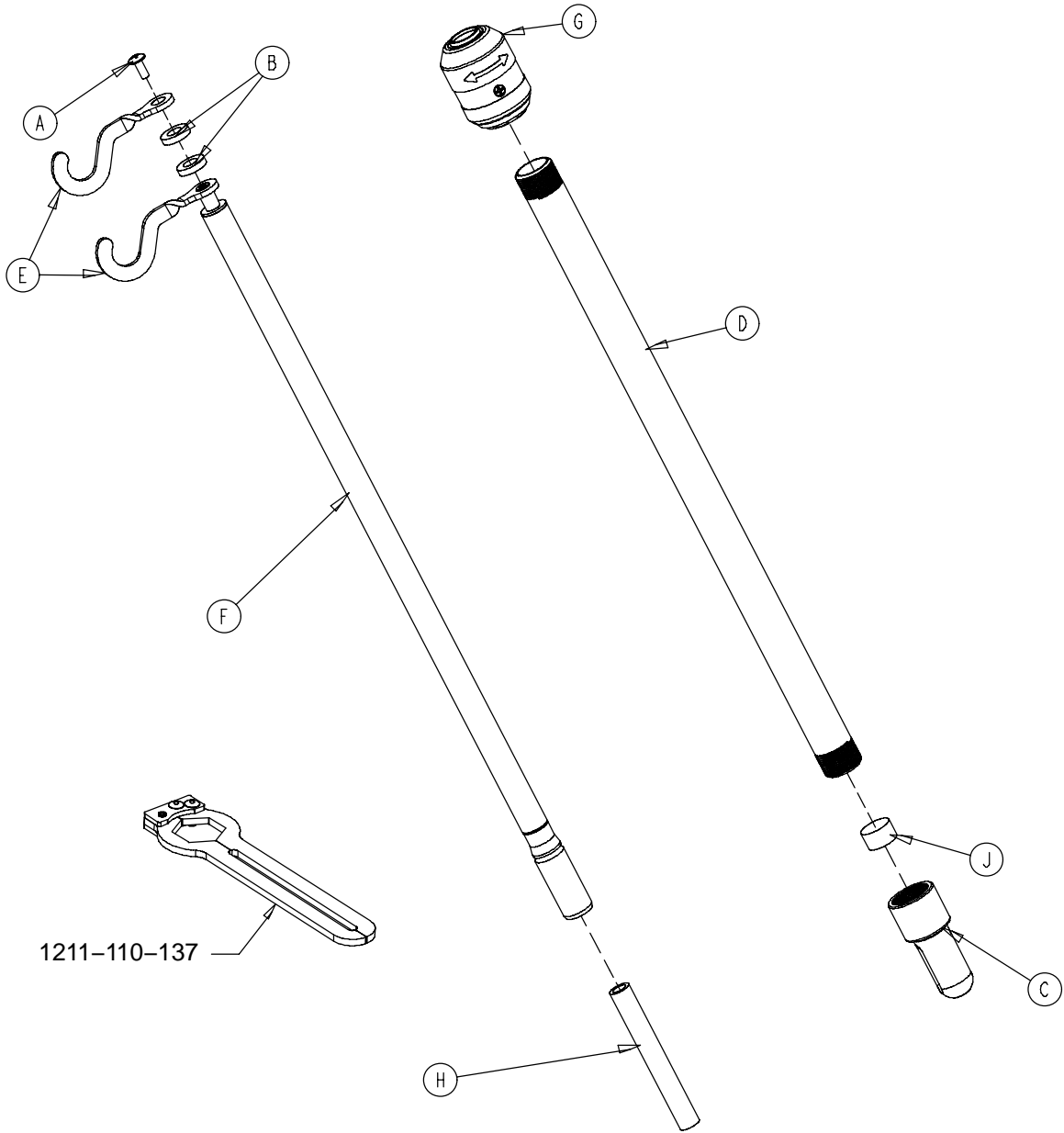

A	0007-040-000	1
B	0052-017-000	2
C	0785-035-003	1
D	0785-035-027	1
E	1010-059-016	2
F	0785-035-102	1
G	0785-035-103	1
H	0785-035-301	1
J	1001-259-013	1

0785-035-642/0785-035-637/0785-035-644/0785-035-639

0785-035-342/0785-035-337/0785-035-344/0785-035-339

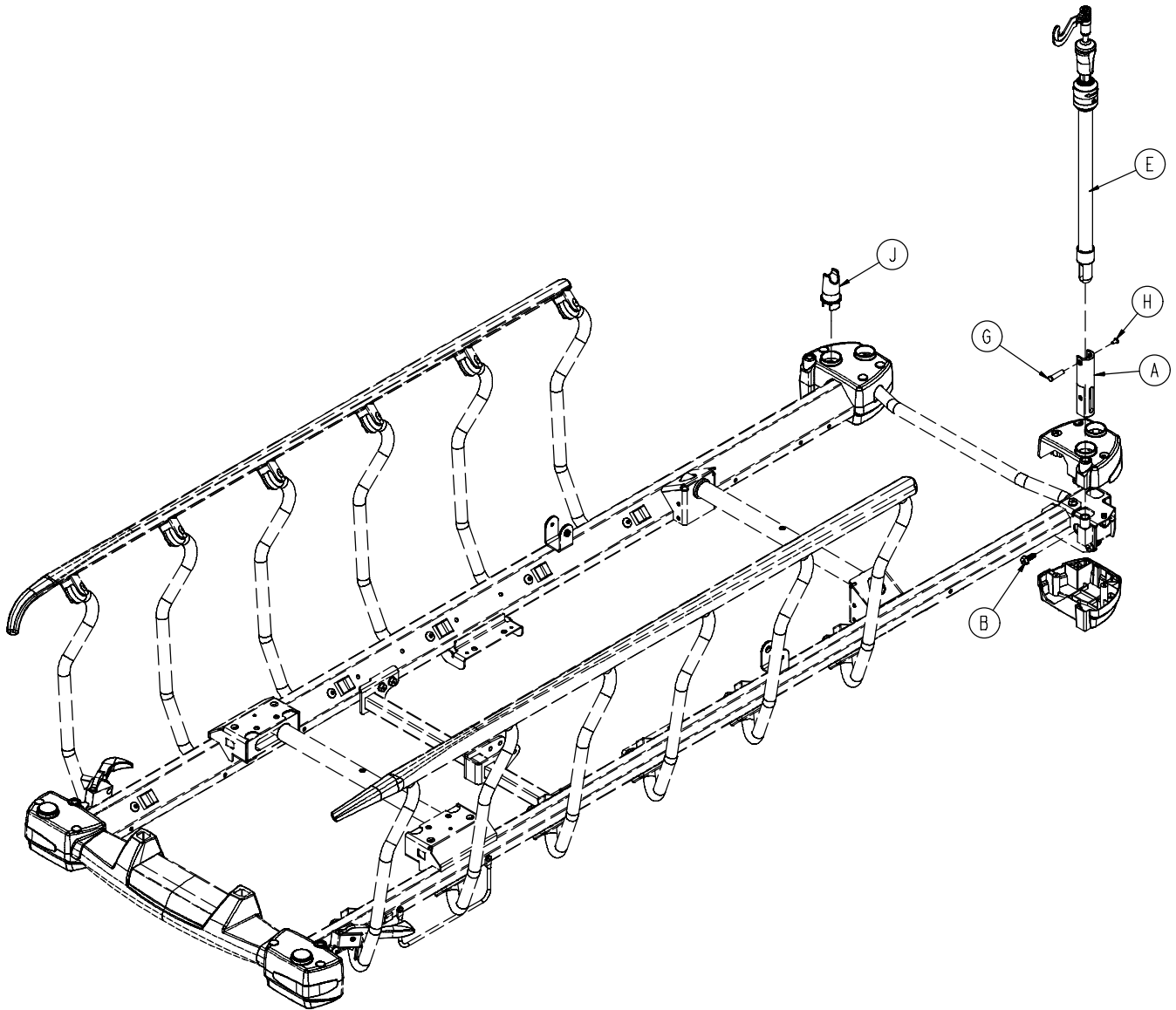


0785-035-642 / 0785-035-637 / 0785-035-644 / 0785-035-639

A	0785-010-020	1
B	0023-288-000	1
C	0785-035-200	1
G	0785-048-025	1
H	0025-079-000	1
J	0785-035-004	1

0785-035-342 / 0785-035-337 / 0785-035-344 / 0785-035-339


A	0785-010-020	1
B	0023-288-000	1
C	0785-035-300	1
G	0785-048-025	1
H	0025-079-000	1
J	0785-035-004	1

0785-035-200

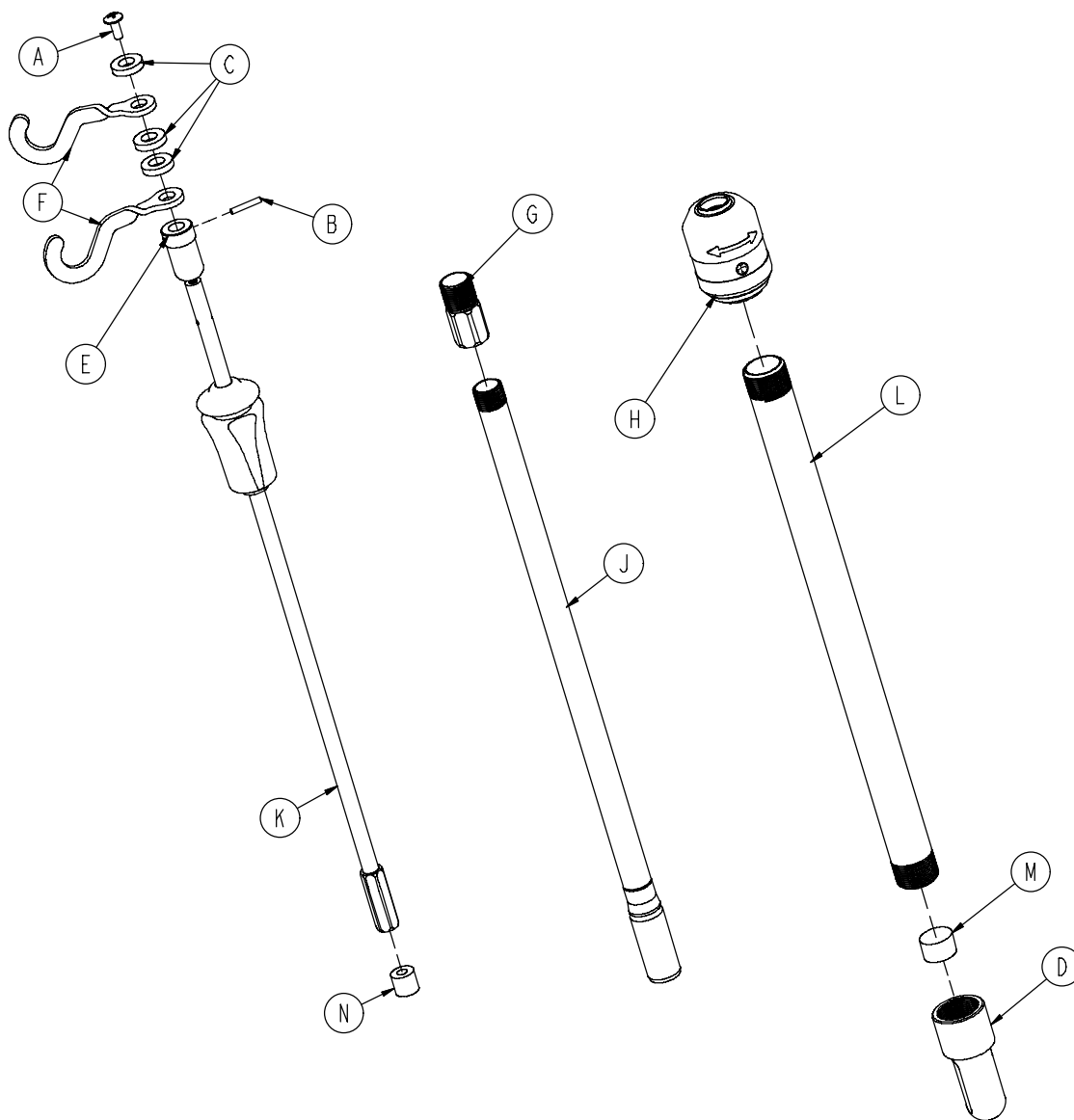


A	0007-004-000	1
B	0026-076-000	1
C	0052-017-000	3
D	0785-035-003	1
E	0785-035-026	1
F	1010-059-016	2
G	1211-110-016	1
H	0785-035-103	1
J	0785-035-201	1
K	0785-035-202	1
L	0785-035-204	1
M	1001-259-013	1
N	1001-359-014	1

0785-035-300

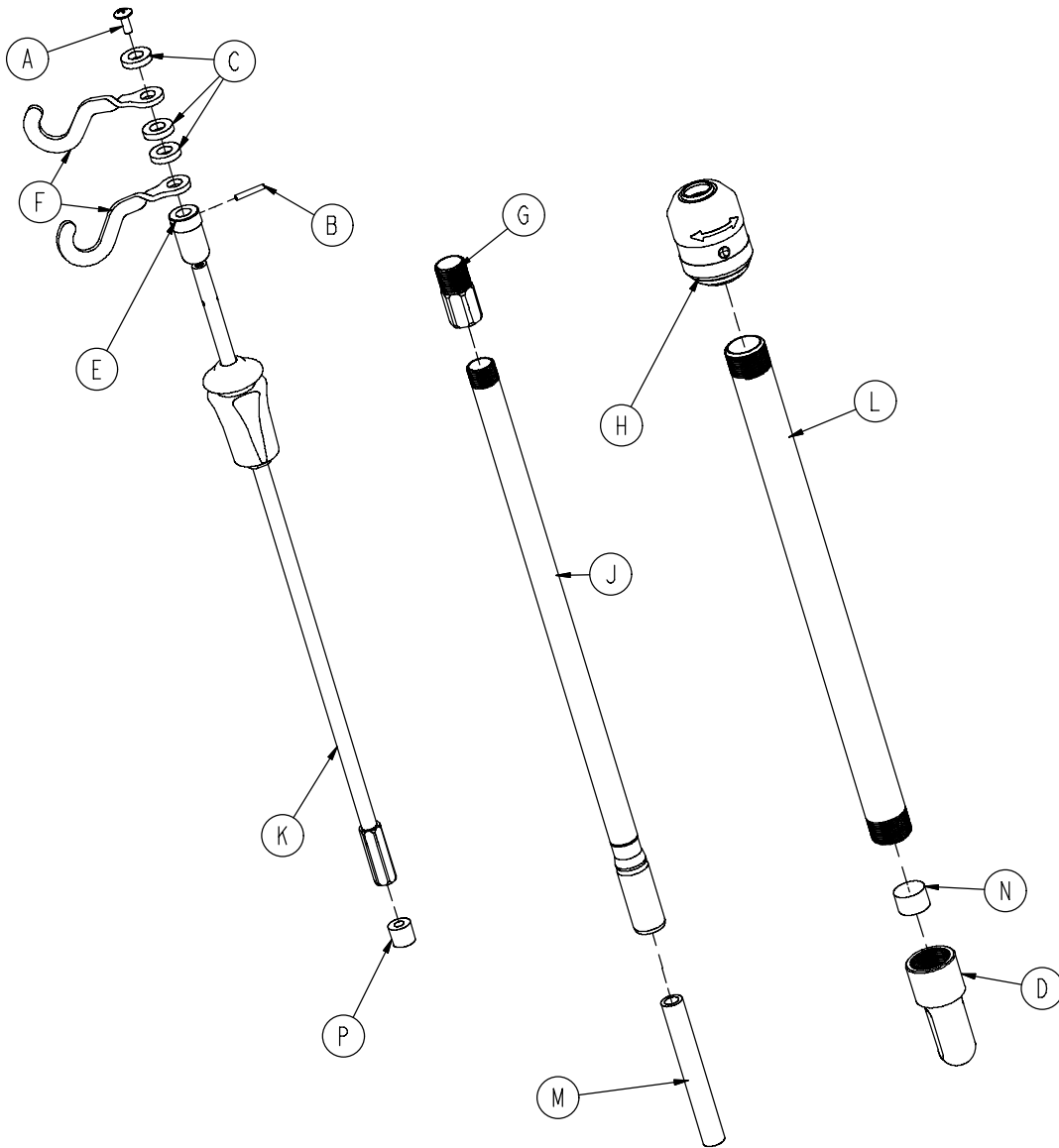


A	0007-004-000	1
B	0026-076-000	1
C	0052-017-000	3
D	0785-035-003	1
E	0785-035-026	1
F	1010-059-016	2
G	1211-110-016	1
H	0785-035-103	1
J	0785-035-201	1
K	0785-035-202	1
L	0785-035-204	1
M	0785-035-301	1
N	1001-259-013	1
P	1001-359-014	1

0785-035-202

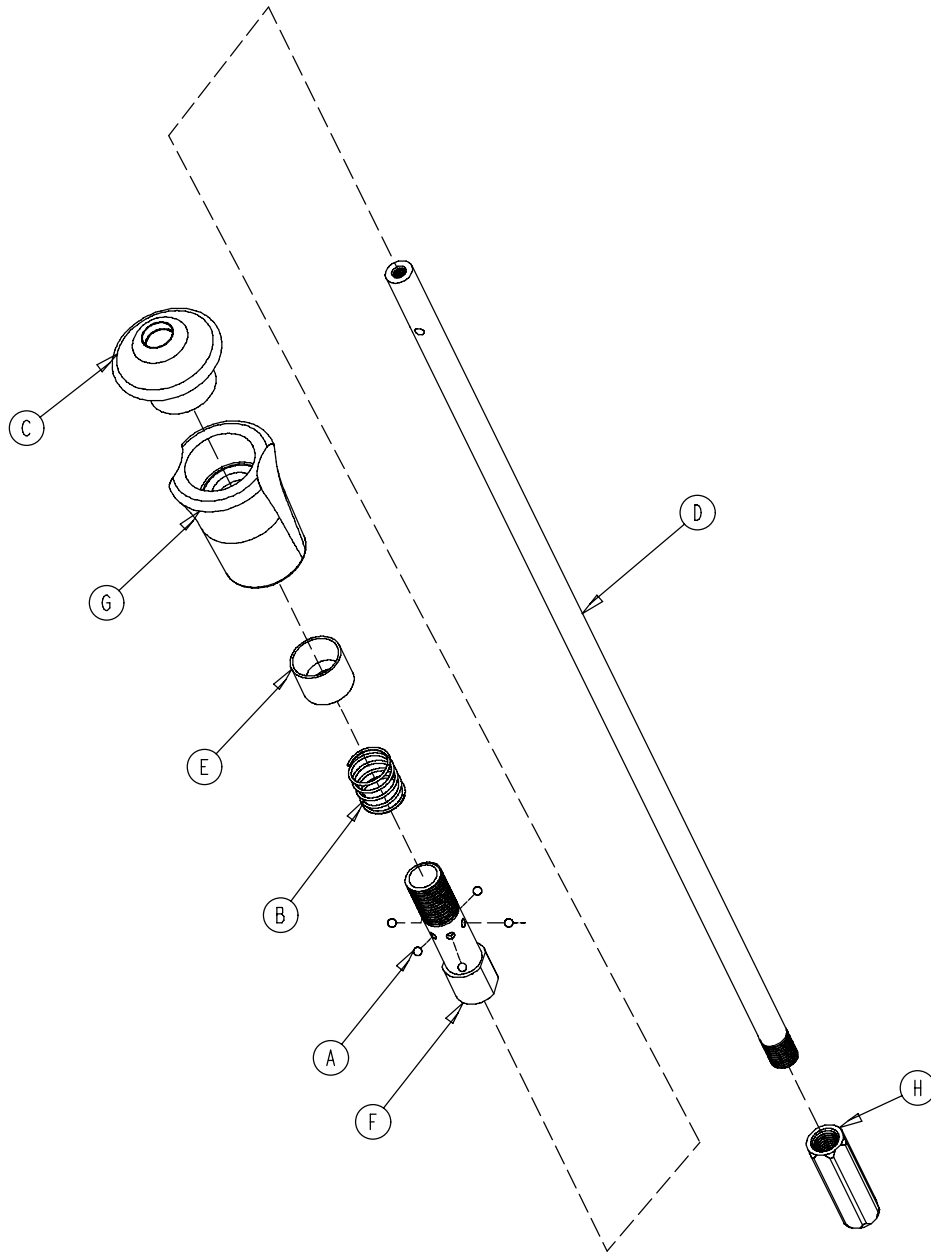

A	0031-021-000	6
B	0038-303-000	1
C	0785-035-037	1
D	0785-035-206	1
E	1010-061-013	1
F	1010-061-016	1
G	1010-061-018	1
H	1211-110-017	1

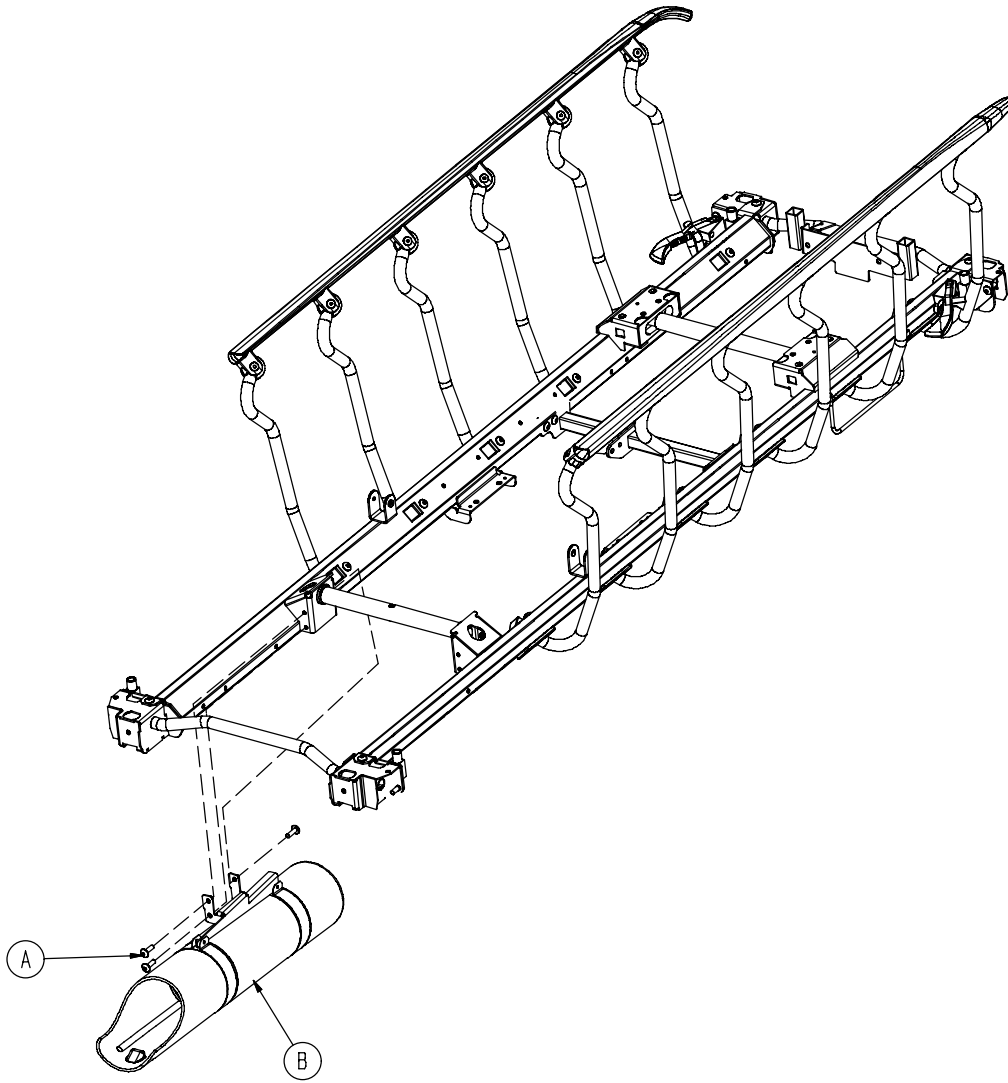


A
B

0023-288-000
0785-030-601

3
1

0785-030-310

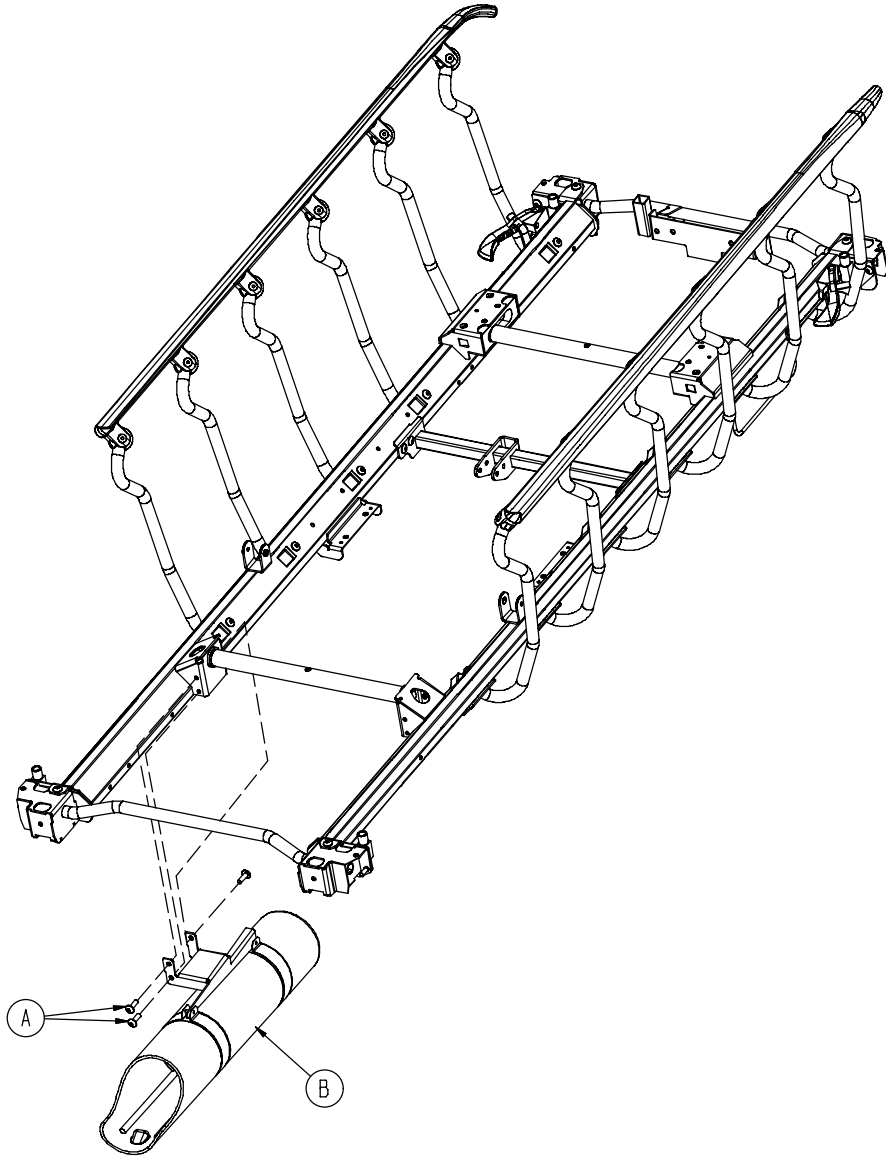


A
B

0023-288-000
0785-030-301


3
1

0785-030-601

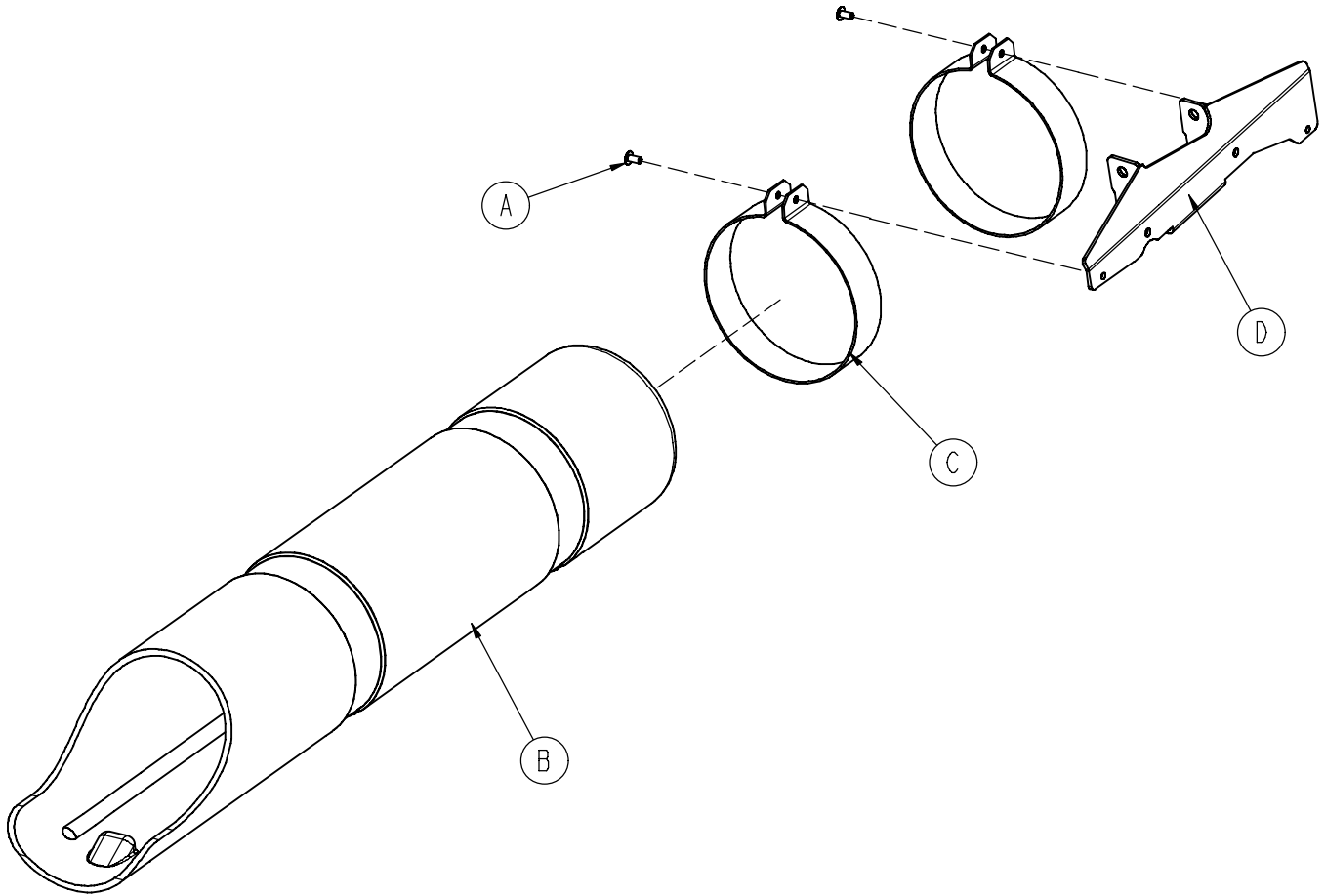


A	0025-079-000	2
B	0785-030-009	1
C	0785-030-021	2
D	0785-130-608	1

0785-030-301

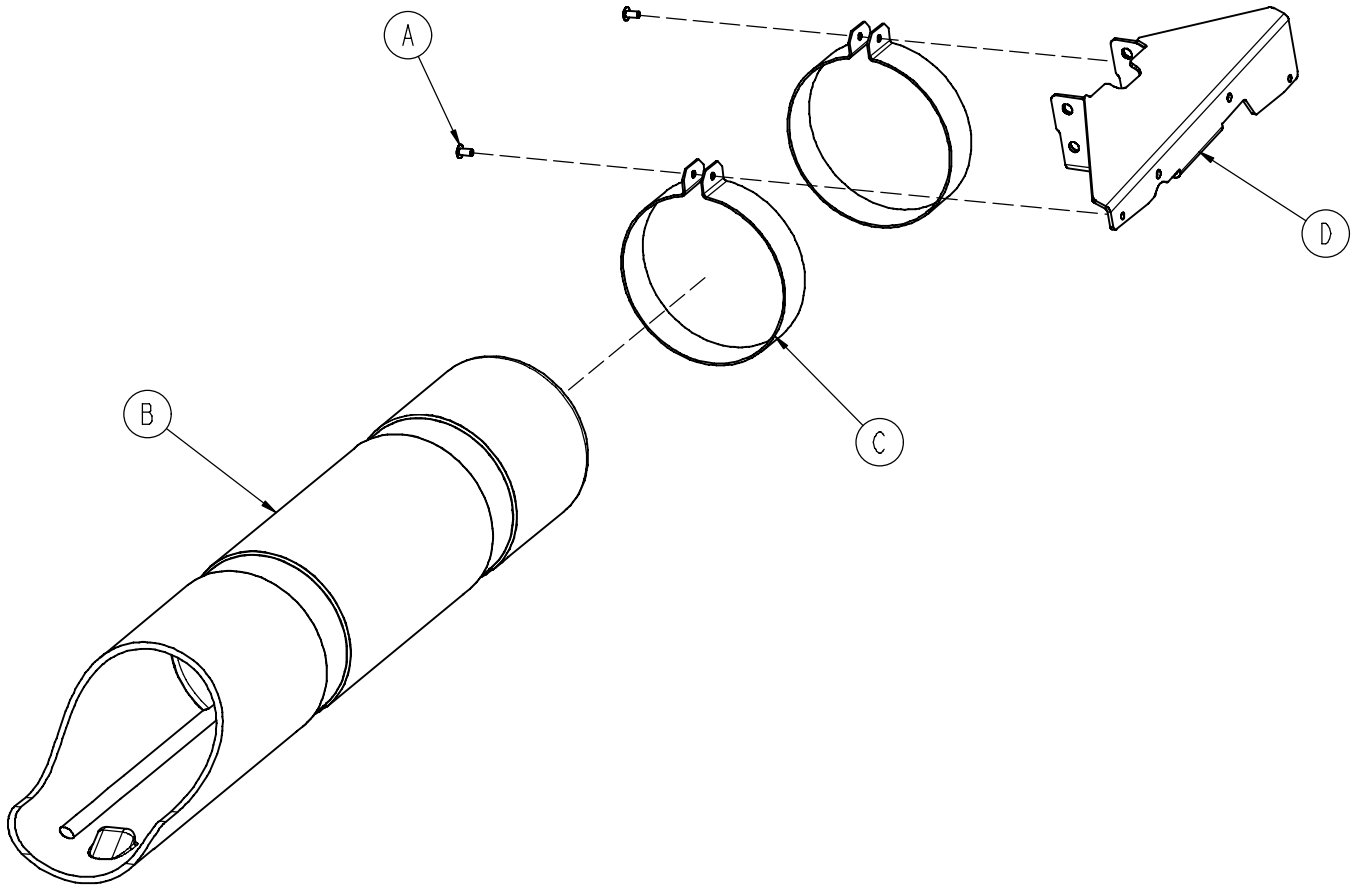


A	0025-079-000	2
B	0785-030-009	1
C	0785-030-021	2
D	0785-130-308	1

1020-130-000

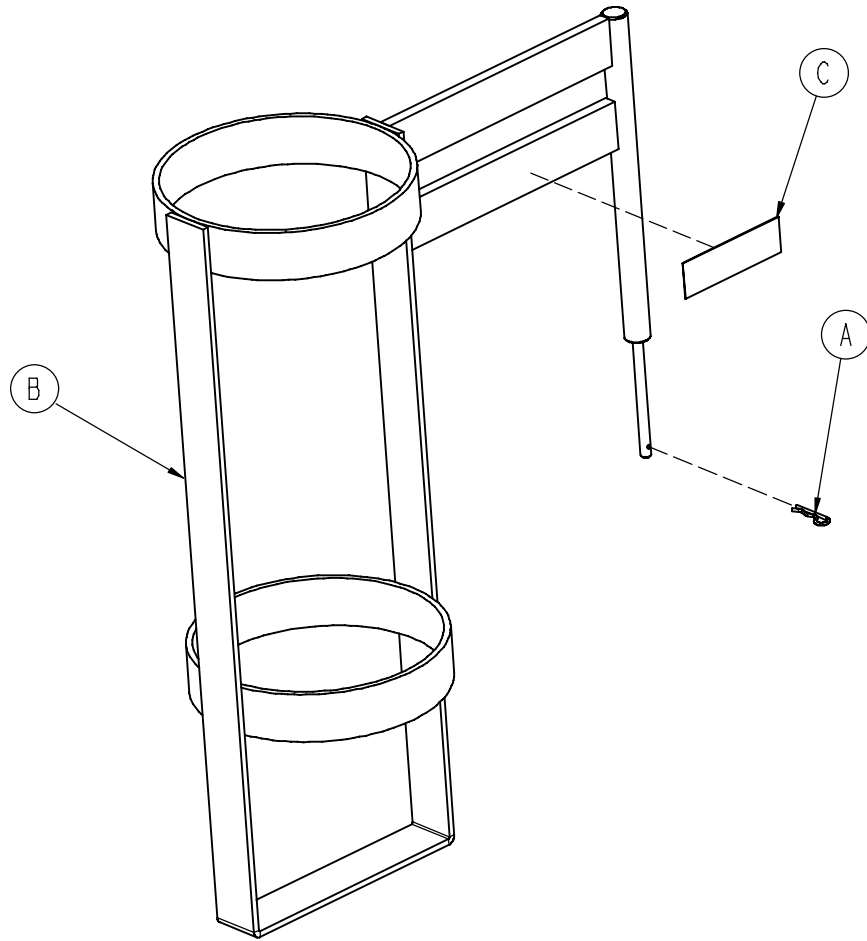


A	0027-030-000	1
B	1020-030-111	1
C	1020-030-117	S1

0785-045-200

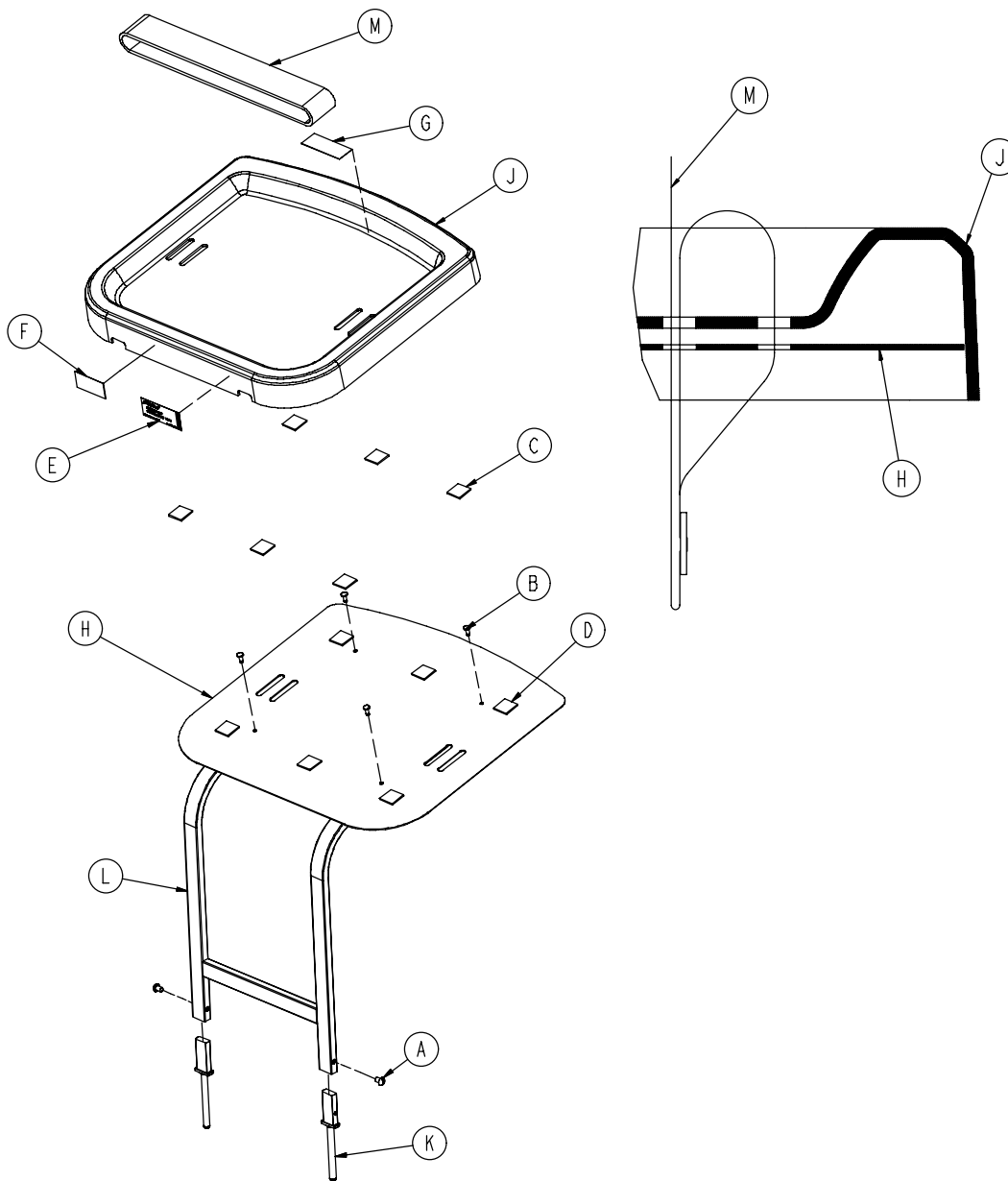


A	0002-044-000	2
B	0025-055-000	4
C	0029-008-000	6
D	0029-010-000	6
E	0785-045-201	1
F	0785-045-402	1
G	0785-045-403	1
H	0785-045-204	1
J	0785-045-207	1
K	0785-045-208	1
L	0785-045-210	1
M	1010-050-021	1

0785-045-400

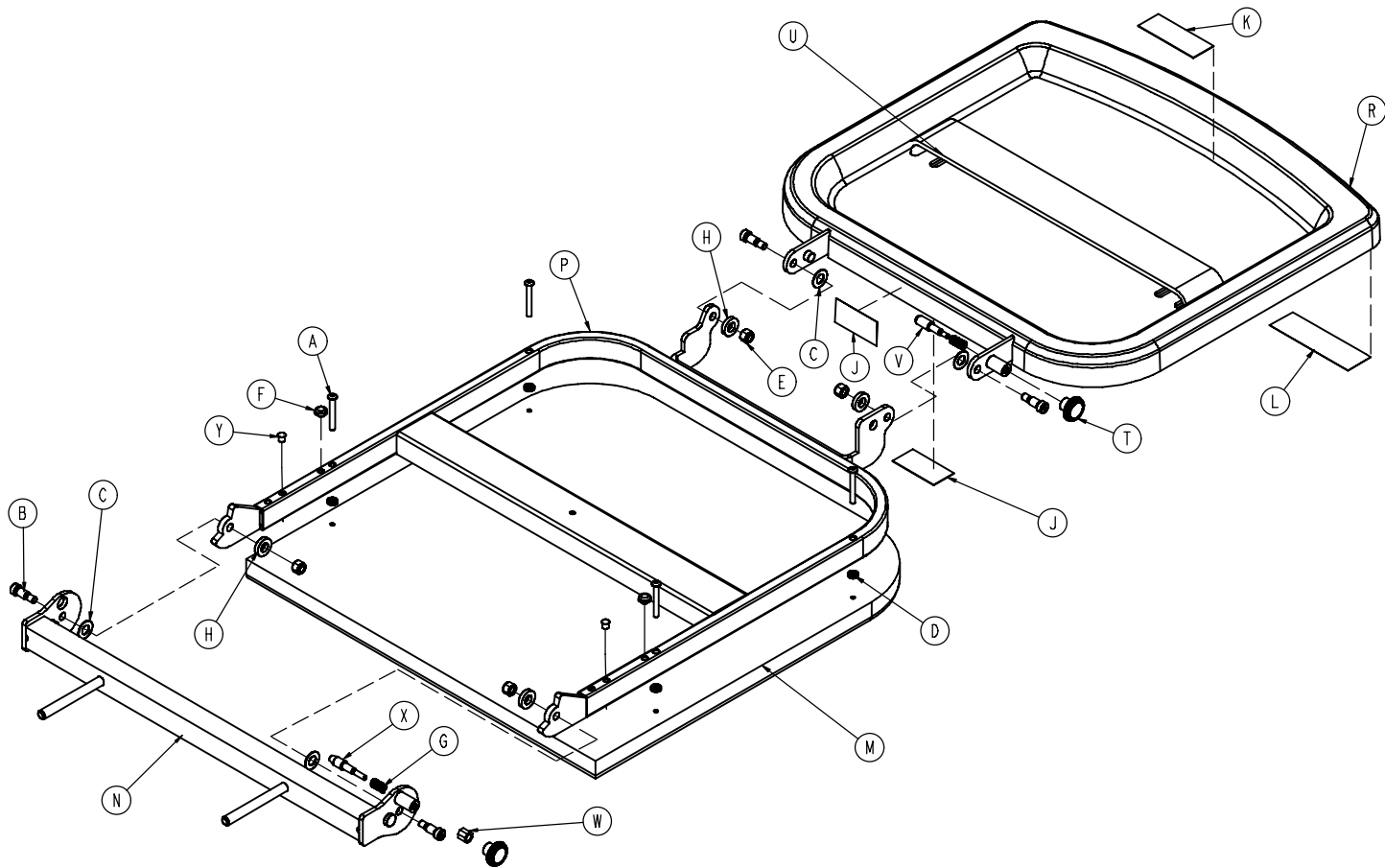


A	0004-215-000	4
B	0008-049-000	4
C	0014-020-000	4
D	0014-120-000	4
E	0016-028-000	4
F	0037-052-000	2
G	0038-133-000	2
H	0052-017-000	4
J	0785-045-402	2
K	0785-045-403	1
L	0785-045-401	1
M	0785-045-405	1
N	0785-045-410	1
P	0785-045-420	1
R	0785-045-430	1
T	0785-045-450	2
U	1010-050-021	1
V	1010-050-242	1
W	1010-050-250	1
X	1010-050-248	1
Y	0721-031-065	2

0785-045-700

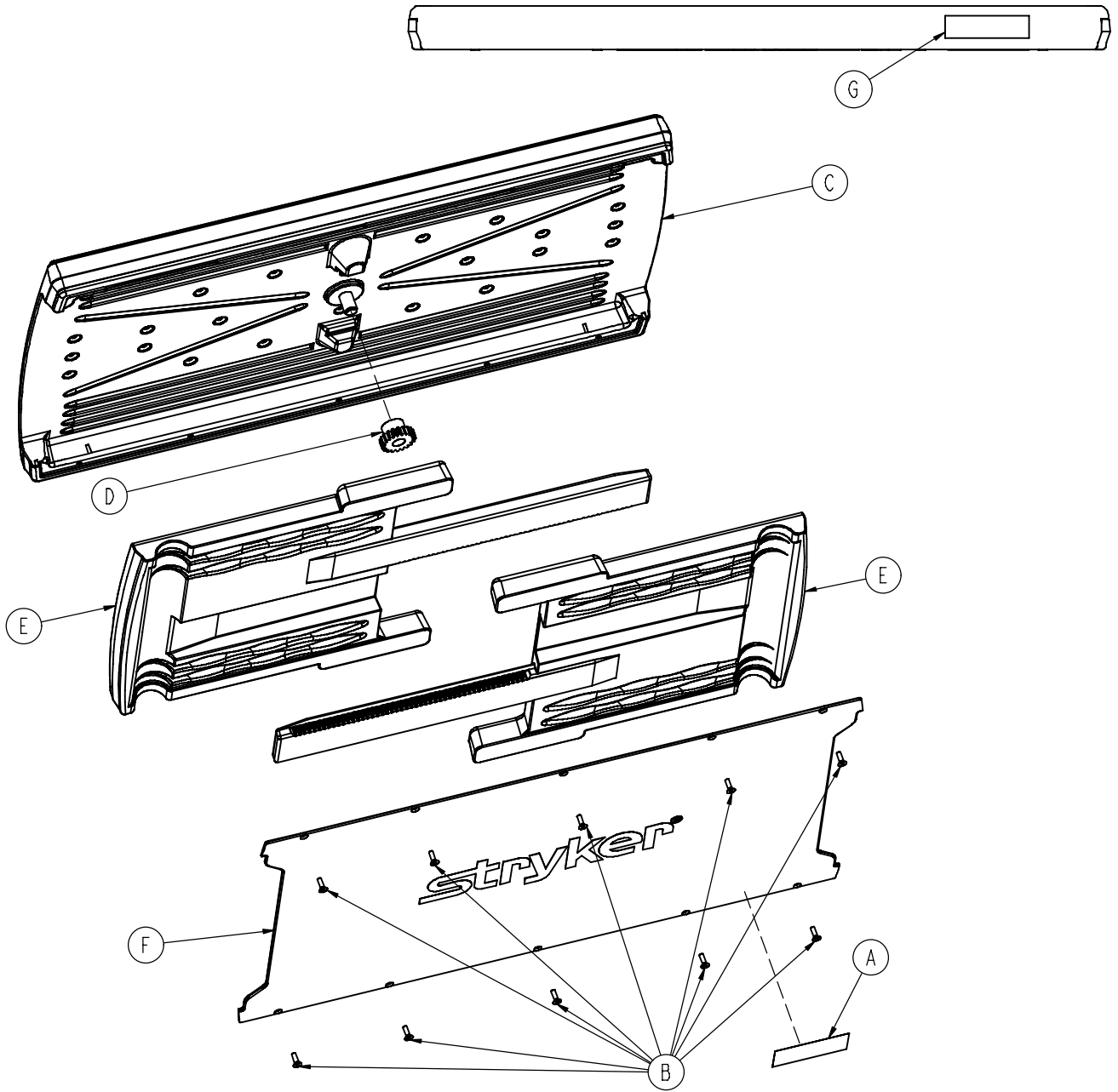


A	0785-045-701	1
B	0785-045-704	10
C	0785-045-705	1
D	0785-045-706	1
E	0785-045-710	2
F	0785-045-715	1
G	0785-045-403	1

0785-045-800

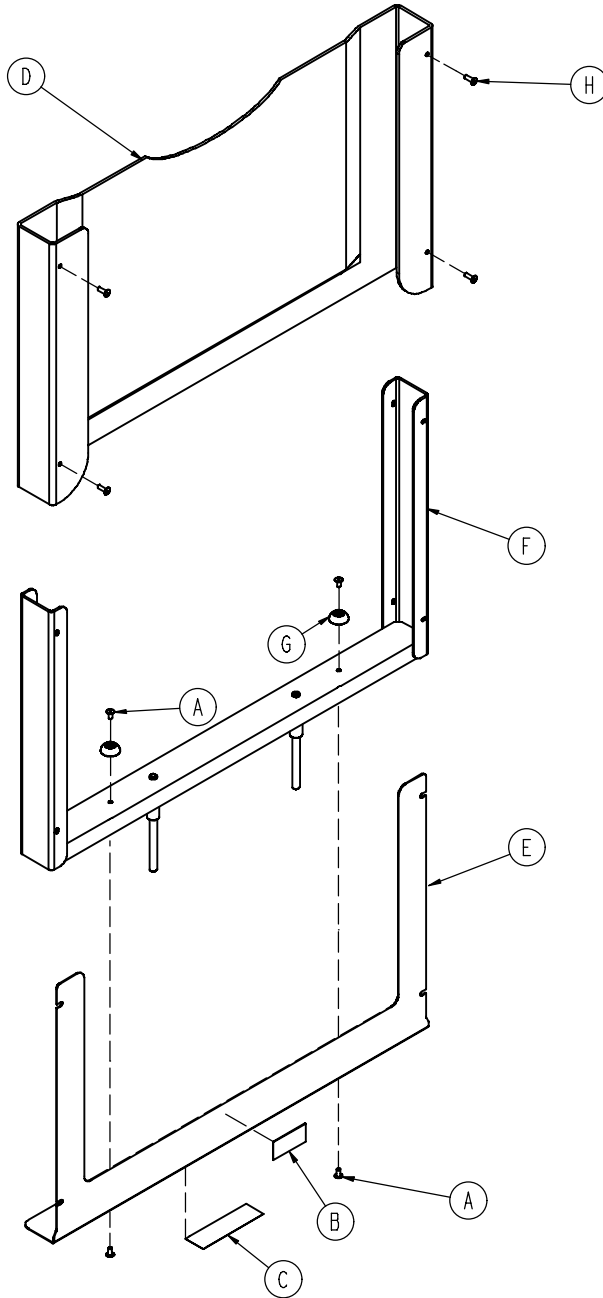

A	0025-079-000	4
B	0785-045-402	1
C	0785-045-801	1
D	0785-045-805	1
E	0785-045-806	1
F	0785-045-810	1
G	0946-001-155	2
H	0025-115-000	4

0785-045-500

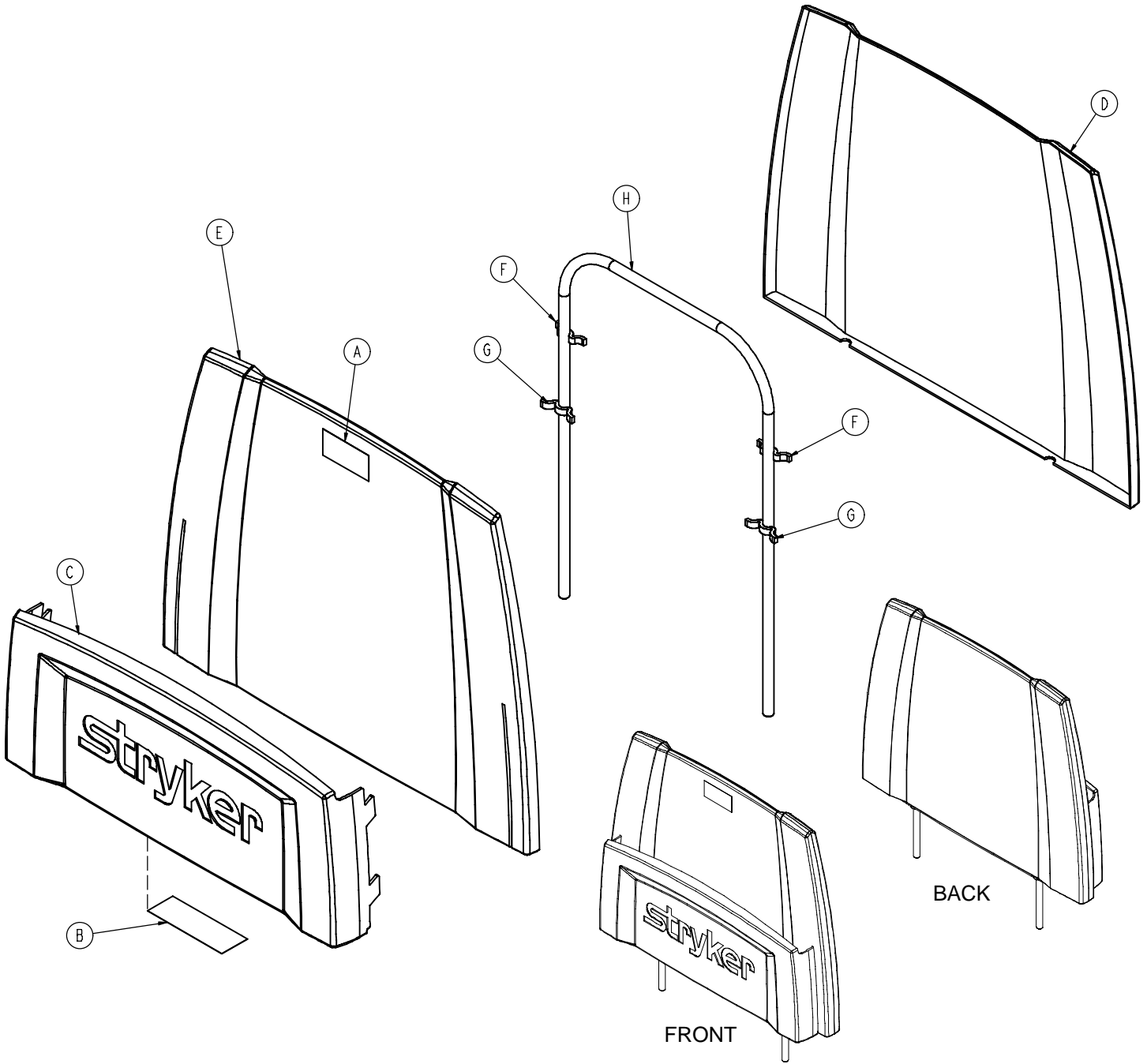

A	0785-045-402	1
B	0785-045-501	1
C	0785-045-511	1
D	0785-045-512	1
E	0785-045-513	1
F	0785-045-514	2
G	0785-045-515	2
H	0946-029-010	1

European Representative

Stryker EMEA RA/QA Director
Stryker France
ZAC Satolas Green Pusignan
Av. De Satolas Green
69881 MEYZIEU Cedex
France

stryker[®]

3800 E. Centre Ave., Portage, MI 49002
(800) 327-0770
www.stryker.com



JH 8/06 1015-600-002 REV D