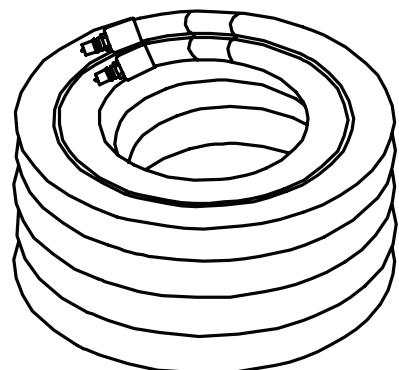
**Altrix® Precision Temperature Management System**

- Insulated Colder Connector® Hose
- Manguera Colder Connector® aislada
- Isolierter Colder Connector®-Schlauch
- Tuyau isolé Colder Connector®
- Tubo isolato Colder Connector®
- Geïsoleerde Colder Connector®-slang
- Isolerad Colder Connector®-slang
- Isoleret Colder Connector® slang
- Mangueira Colder Connector® isolada
- Isolert Colder Connector®- tilkoblingsslange
- Izolowany giętki przewód Colder Connector®
- Изолированный соединительный шланг Colder Connector®
- Yalıtımlı Colder Connector® Hortumu
- Izolovaná hadice Colder Connector®
- 絶縁Colder Connector®ホース
- 隔热 Colder Connector® 软管
- 절연 Colder Connector® 호스

**REF** 8001-064-135

- Operations Manual
- Manual de uso
- Bedienungsanleitung
- Manuel d'utilisation
- Manuale d'uso
- Bedieningshandleiding
- Användarhandbok
- Betjeningsmanual
- Manual de utilização
- Brugerhåndbok
- Podręcznik użytkownika
- Руководство по эксплуатации
- Kullanma Kılavuzu
- Příručka pro obsluhu
- 操作マニュアル
- 操作手册
- 작동 설명서

**Introduction**

This manual assists you with the use of the Stryker Model 8001-064-135 **Insulated Colder Connector Hose**. Read this manual before operating this product and keep a copy on file. Set methods and procedures to educate and train your staff on the safe operation of this product. For cleaning, disinfecting, or more information, see the **Altrix® Precision Temperature Management System Operations Manual**.

**Product description**

The **Insulated Colder Connector Hose** provides an interface from the thermal transfer devices with **Colder Connector** connections to the controller.

**Expected life**

The **Insulated Colder Connector Hose** has a one year use expected life under normal use conditions.

**Disposal/recycle**

Always follow the current local recommendations and/or regulations governing environmental protection and the risks associated with recycling or disposing of the equipment at the end of its useful life.

**Specifications**

Patient facing material	EPDM (Ethylene Propylene DI-Monomer) Sponge Extrusion; Flame Retardant Grey Flexible PVC; Adhesive Lined, Polyolefin; Polypropylene	
Length	10 ft	3.05 m

Stryker reserves the right to change specifications without notice.

In accordance with the European REACH regulation, this article contains bis(2-ethylhexyl) phthalate.

**Contact information**

Contact Stryker Customer Service or Technical Support at 1-800-327-0770. To view your operations or maintenance manual online, see <https://techweb.stryker.com/>.

Have the Lot batch code available when you call Customer Service. Include the Lot batch code in all written communication.

**Note** - The user and/or the patient should report any serious product-related incident to both the manufacturer and the Competent authority of the European Member State where the user and/or patient is established.

**Symbols**

- General warning
- Caution
- Refer to instruction manual/ booklet
- Consult instructions for use
- Temperature controller
- Quantity
- Batch code
- Catalogue number
- CE mark
- Manufacturer
- European medical device
- Authorized representative in the European Community

**CAUTION**  
US Federal Law restricts this device to sale by or on the order of a licensed physician.

**Setup**

Connect the **Insulated Colder Connector Hose** to the controller (A) hose connection port. Connect the **Colder Connector** end (B) of the hose to the thermal transfer device (C) (Figure 1).

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